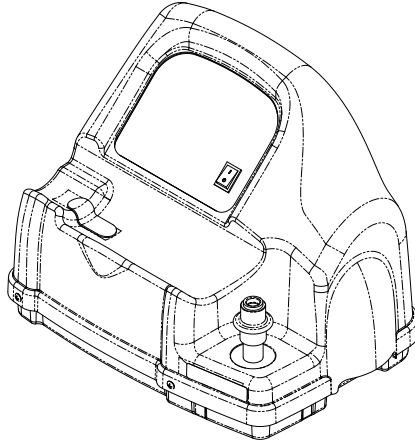
Operator Manual

HomeFill® II Compressor

Model IOH200AW

English UK.....	2
Dutch	39
French	77
Greek	120
Italian.....	163
Portuguese	206
Spanish	246

User: Before using the product, read this manual and save for future reference.

Supplier: This manual must be given to the end user.

For more information regarding Invacare products, parts, and services, please visit www.invacare.eu.com

Yes, you can.

⚠ WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE DANGERS, WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, SERIOUS INJURY OR PROPERTY DAMAGE MAY RESULT.

⚠ DANGER

Users MUST NOT SMOKE while using this device. Do not keep matches, lit cigarettes or other sources of ignition in the same room as this product. Textiles and other materials which would are not normally flammable are easily ignited and burn with great intensity in oxygen enriched air. Failure to observe this warning can result in severe fire, property damage and cause physical injury or DEATH.

NOTE: Updated versions of this manual are available at www.invacare.com.

TABLE OF CONTENTS

SPECIAL NOTES	4
TYPICAL PRODUCT PARAMETERS	6
LABEL LOCATION	8
DEALER INFORMATION	9
HOMEFILL II INFORMATION	10
SECTION 1—GENERAL GUIDELINES	11
SECTION 2—INITIAL SETUP	15
Initial Set Up of Compressor	15
SECTION 3—CHECKLIST AND CYLINDER INSPECTION.....	19
Compressor Operation Checklist	19
Cylinder Prefill Inspection	19
SECTION 4—COMPRESSOR OPERATION.....	22
Connecting/Disconnecting the Cylinder to/from the Compressor	22
Turning the Compressor On	27
Setting the Flow Knob to the Prescribed Setting	28
Technical Description	29
SECTION 5—INDICATOR LIGHTS	31
SECTION 6—TROUBLESHOOTING/ CYLINDER FILL TIMES	33
SECTION 7—MAINTENANCE.....	36
Cleaning the Compressor Filter	36
Cabinet	37

SPECIAL NOTES

Check all parts for shipping damage. In the event of damage, DO NOT use. Contact Invacare for further instruction.

Signal words are used throughout this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the following table for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, could result in property damage.

ACCESSORIES

Many different types of humidifiers, oxygen tubing, cannulas and masks can be used in conjunction with this device. You should contact your local home care provider for advice on which of these devices is most suitable for you. Your home care provider can also provide advice on proper usage, maintenance, and cleaning.

WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

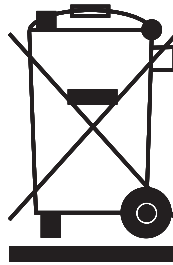
RECYCLING INFORMATION

This product is supplied by an environmentally aware manufacturer which adheres to the Waste Electrical and Electronic Equipment (WEEE) Directive 2002/96/CE.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) which are not deemed appropriate by current legislation.

A 'crossed out wheelie bin' symbol has been placed on this product to encourage you to recycle wherever possible.

Please be environmentally responsible and recycle this product through your recycling facility at the end of its useful life.

TYPICAL PRODUCT PARAMETERS

Regulatory Listing:	EN/IEC 60601-1-2 EMC
	Alternating Current
	Type BF equipment
I	Unit running
O	Unit not running
	Attention - Consider Accompanying Documents
	DO NOT smoke
	Class II, Double Insulation
CE	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

TYPICAL PRODUCT PARAMETERS - COMPRESSOR

Operating temperature:	50 to 95° F (10 to 35° C) @ 20 to 60% non-condensing humidity
Storage temperature:	-10 to 150° F (-23 to 65° C) @ 15 to 95% humidity
Input Pressure Required:	14 - 21 PSI (96.5-144.7 kPa)
Input Flow Required:	2 L/min
Input O₂% Required:	>90% O ₂
Width:	20 ¼ inches (51.44 cm)
Height:	15 inches (38.1 cm)

Depth:	16 inches (40.64 cm)
Weight:	33 lbs (14.97 kg)
Shipping Weight:	39 lbs (17.69 kg)

CONSERVER/REGULATOR SPECIFICATIONS

Oxygen Output	Settings 1 - 5 L/min pulse 2 L/min. continuous
----------------------	---

LABEL LOCATION

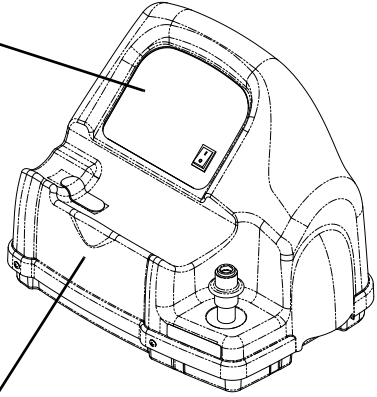
DANGER - RISK OF ELECTRIC SHOCK

DO NOT remove cover. Refer servicing to qualified service personnel.

DANGER- RISK OF FIRE

NO SMOKING - Keep **ALL** sources of ignition out of the room in which this product is located and away from areas where oxygen is being delivered. Textiles, oil and other combustibles are easily ignited and will burn with great intensity in oxygen enriched air.

P/N 1145730 Rev A - 4/18/07

WARNING

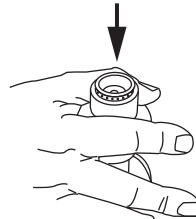
Keep all connections free of oil and grease. See Owner's Manual before cleaning. **NEVER** oil or lubricate coupling.

Cylinders filled on this unit **MUST** be labeled "Contents of this cylinder are the product of an oxygen concentrator."


Keep all connections free of oil and grease to avoid violent ignition. See Owner's Manual before cleaning.

This unit contains moving and high pressure components. **Manufacturer Service Only.**

BEFORE COUPLING CYLINDER, PUSH DOWN ON SLEEVE UNTIL GREEN DOTS ARE VISIBLE.

P/N 1048080 Rev A -4/11/07

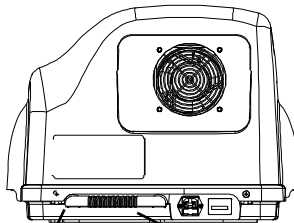Serial Number
Label Location

Specification
Label Location

DEALER INFORMATION

NOTE: Invacare recommends leaving a full cylinder of oxygen with the patient after setting up the HomeFill II compressor.

Before leaving a HomeFill II Compressor with a user, the following checklist **MUST** be completed:

- Make sure the pressure test on cylinder has not expired.
- Instruct the user on the safe operation of the concentrator and review all warnings.
- Instruct the user on the safe operation of the HomeFill II compressor and review all warnings.
- Leave a full cylinder of oxygen with the user.
- Leave a copy of all warnings and filling instructions with the user.

HOMEFILL II INFORMATION

HomeFill II is an accessory device to the oxygen concentrator, which enables the patient to refill custom portable oxygen cylinders for personal use.

NOTICE


The HomeFill II compressor is for use with specially designed Invacare Platinum™ 5 and Platinum 9 HomeFill II compatible oxygen concentrators.

The oxygen concentrator can be set to maximum flow rate of when operating with HomeFill II compressor:

Platinum 5 - 2.5 L/min. or less

Platinum 9 - 5 L/min. or less

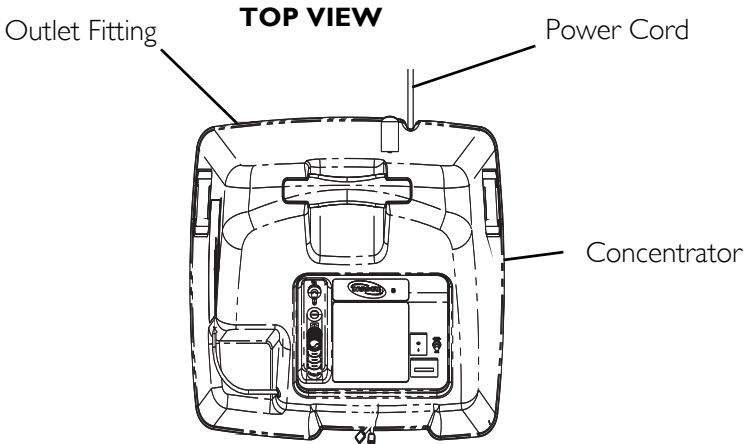To determine if your oxygen concentrator is HomeFill II compatible, examine the lower right corner of the rear of the oxygen concentrator for an outlet fitting and refer to the illustration below.

SECTION I—GENERAL GUIDELINES

WARNING

SECTION I - GENERAL GUIDELINES contains important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first reading and fully understanding these instructions and any additional instructional material such as **Owner's Manuals, Service Manuals or Instruction Sheets** supplied with this product or optional equipment. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a health care professional, dealer or technician before attempting to use this equipment - otherwise, injury or damage may occur.

In order to ensure the safe installation, assembly and operation of the HomeFill II Compressor these instructions **MUST** be followed.

“Caution: Statutory law can restrict this device to sale by or on order of a doctor, or any other practitioner licensed by the law of the governmental agency in which he/she practices to use or order the use of this device.”

WARNING

This device is for the sole use of the oxygen patient. Cylinders **MUST ONLY** be used by the oxygen patient and may not to be distributed to any other individual for any purpose.

The maximum patient flow while filling the cylinders with the IOH200AW is 2.5 L/min. for Platinum 5 and 5 L/min. for Platinum 9. Patients with a higher prescribed flow should not transfill while receiving supplemental oxygen treatment from the Platinum Concentrator.

⚠ WARNING (CONTINUED)

NEVER block product air openings or place the product on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like.

DO NOT use outdoors.

NEVER use near any type of flame or flammable/explosive substance, vapour or atmosphere.

DO NOT disassemble. The unit is provided with double insulation and has no serviceable parts. Should any service be required, contact Invacare Technical Service.

If the compressor has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or immersed in water, please call a qualified technician for inspection and repair.

If any leakage of the oxygen cylinder is detected DO NOT attempt to use the cylinder. Turn the flow selector knob on the regulator to "Off". If leakage persists, place the cylinder outdoors and notify your Home Health Care Provider or Service Representative of this condition.

DO NOT allow oil from your hands or other sources to come into contact with the compressor coupling or bottle fittings. In the presence of oxygen, these substances may become flammable and may cause injury. Avoid touching the bottle fitting whenever possible. Some examples of items to be avoided are hand creams and lotions, cooking oils, suntan oils or sunblocks and similar oil-based products. If you use these types of products Invacare suggests you wash your hands prior to using the HomeFill unit. Should the bottle fitting come into contact with any such substance it should be wiped with a damp cloth prior to being connected to the compressor.

⚠ WARNING (CONTINUED)

NEVER oil or lubricate the compressor coupler or bottle fittings. NEVER use any penetrating oil or lubricant such as WD-40 or 3 in One. If you experience connection difficulties, please verify that the bottle fitting has been inserted straight down into the compressor coupler. In the event of continued difficulties, contact your supplier for assistance.

Keep the oxygen tubing, cord, and unit away from heated or hot surfaces, including space heaters, blankets, cookers and similar electrical appliances.

The HomeFill II is equipped with a high pressure relief valve to ensure user safety. When activated, this safety feature will emit an extremely loud noise. Should this noise occur, turn the unit Off and contact your Home Health Care Provider or Service Representative.

Before moving or repositioning either the compressor or concentrator, ALWAYS push the power switch to the Off (O) position and disconnect the AC power cords and the interconnect hose between the compressor and concentrator. Failure to follow this procedure may result in damage to the unit or personal injury.

Children should ALWAYS be supervised around the HomeFill II unit. When used with the HomeFill table or similar elevated support surfaces, a child could knock over the table or other surface thus causing the unit to fall. Failure to observe this warning may result in personal injury or product damage.

Handling

Use extreme care when handling and filling an oxygen cylinder. Full oxygen cylinders are pressurised and can turn into projectiles if dropped or mishandled.

⚠ WARNING (CONTINUED)

NEVER transport a compressor with a cylinder connected to or stored on the compressor, otherwise injury or damage can occur.

DO NOT store filled oxygen bottles near a furnace, hot water tank, in the trunk of a car or in other areas of high temperature. Storage in such areas may cause the bottle to explode or can result in fire.

SECTION 2—INITIAL SETUP

NOTE: Suppliers Only - Refer to Dealer Information on page 9 prior to performing this procedure.

Initial Set Up of Compressor

Compressor Initial Setup Checklist

Before operating the HomeFill II Compressor for the first time, complete the following checklist:

- Choose a location for the compressor.
- Transport the compressor to the desired location.
- Connect the power cord to the compressor.
- Connect the compressor to the concentrator.

Choosing a location

⚠ WARNING

NEVER use near any type of flame or flammable/explosive substance, vapour or atmosphere.

Keep the oxygen tubing, cord, and unit away from heated or hot surfaces, including space heaters, blankets, cookers and similar electrical appliances.

The compressor should be at least three inches away from walls, curtains or furniture to ensure sufficient air flow.

DO NOT use inside a cupboard.

The compressor should rest on a hard flat stable surface, such as a table or the Ready Rack.

Compressor Transportation

NOTE: For this procedure, refer to FIGURE 2.1.

⚠ WARNING

NEVER transport a compressor with a cylinder connected to or stored on the compressor as this may result in injury or damage.

CAUTION

When transporting the compressor, take care no to bump the compressor or the connectors into obstacles. Otherwise, damage to the compressor may occur.

NOTE: For this procedure, refer to FIGURE 2.1.

1. There are two methods for the safe transportation of the compressor:
 - A. One hand - Hold the hand grip at the bottom rear of the compressor.
 - B. Both hands - Hold the hand grips on the sides of compressor.

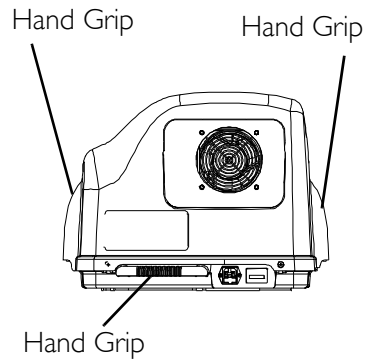
FIGURE 2.1 Compressor Transportation

Installing the Power Cord

NOTE: For this procedure, refer to FIGURE 2.2 on page 17.

1. Ensure the compressor is on a hard flat stable surface, such as a table or Ready Rack.
2. Make sure the power switch is in the Off (O) position.
3. Plug the power cord into the electric connector in the back of the compressor.

-
4. Plug the opposite end of the power cord into the wall socket.

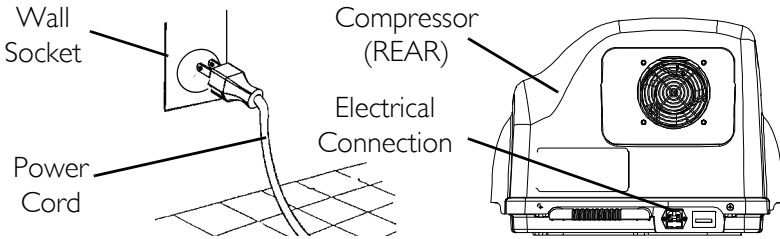
FIGURE 2.2 Installing the Power Cord

Connecting the Compressor to the Concentrator

NOTE: For this procedure, refer to FIGURE 2.3 on page 18.

1. Insert one end of the interconnect hose into the outlet fitting on the back of the concentrator (Detail "A").
2. Insert the opposite end of the interconnect hose into the inlet fitting on the side of the compressor (Detail "B").

NOTE: The metal tabs on the concentrator outlet fitting and the compressor inlet fitting will pop out with an audible "click" when the end of the interconnect hose is properly installed (Detail "C").

3. Loop any excess interconnect hose and secure to the back of the concentrator with the fastening straps.

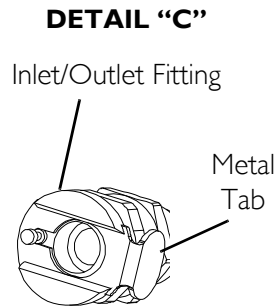
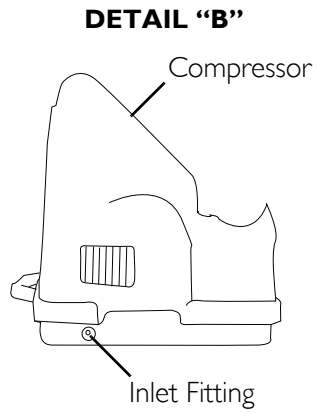
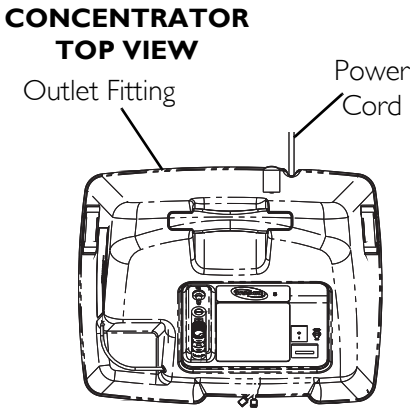
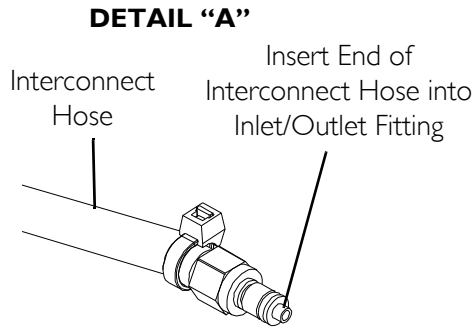
FIGURE 2.3 Connecting the Compressor to the Concentrator

SECTION 3—CHECKLIST AND CYLINDER INSPECTION

Compressor Operation Checklist

Complete the following checklist each time the HomeFill II Compressor is used to fill a cylinder:

- Ensure that the concentrator has been On for at least thirty minutes. Refer to the concentrator Owner's Manual.
- Conduct the prefill inspection on the cylinder.
- Connect the cylinder to the compressor.
- Push the compressor power switch to the On (I) position.
- Examine the indicator lights on the control panel.
- Disconnect and remove the full cylinder.
- Push the compressor power switch to the Off (O) position.
- Repeat this checklist when filling another cylinder.

Cylinder Prefill Inspection

WARNING

ONLY use cylinders that have the Invacare HomeFill II connection and label.

All cylinders MUST be inspected before attempting to fill. Otherwise, injury or damage may occur.

NOTE: For this procedure, refer to FIGURE 3.1 on page 20.

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Authorized Rep. in the EU:
Invacare International Srl
Routte de Cléoux 2
3101 E. La Cour Bay Blvd.
Suresnes, France
Switzerland

0120 9321EC

CE

REF

INVCARE

PN I141470 Rev E

FIGURE 3.1 Cylinder Prefill Inspection

The Hydrostatic Testing Date

Aluminum cylinders **MUST** be tested every five years.

⚠ WARNING

DO NOT fill cylinders that have not been tested in the past five years. **Otherwise, serious personal injury may result. For replacement contact your supplier.**

External Examination

1. Examine the outside of the cylinder for the following conditions, and replace the cylinder in the event of:
 - Dents or surface damage
 - Arc burns
 - Oil or grease
 - Any other signs of damage which may cause the cylinder to be unacceptable or unsafe for use.
2. Inspect the cylinder for evidence of fire or thermal damage. Such evidence includes charring or blistering of the paint or other protective coating or heat sensitive indicator. If fire or thermal damage is found, replace the cylinder.
3. Inspect the cylinder/regulator assembly for the following, and replace if found:
 - Debris, oil or grease
 - Noticeable signs of damage
 - Signs of corrosion inside the valve
 - Signs of excessive heat or fire damage

SECTION 4—COMPRESSOR OPERATION

Connecting/Disconnecting the Cylinder to/from the Compressor

Connecting the Cylinder to the Compressor

⚠ WARNING

NEVER use tools of any kind to connect/disconnect the cylinder and the compressor. Otherwise, severe injury and/or damage may occur.

DO NOT drop oxygen cylinders. Use both hands when handling/transporting oxygen cylinders. Otherwise, injury or damage may occur.

DO NOT use a liquid leak detector to test for leaks.

NOTE: For this procedure, refer to FIGURE 4.1 on page 23 and FIGURE 4.2 on page 24.

1. Examine the cylinder gauge. If the cylinder pressure is **less** than 1500 psig (10342 kPa), proceed to STEP 2 (FIGURE 4.2).

*NOTE: If the cylinder pressure is **greater** than 1500 psig (10342 kPa), DO NOT attempt to top off this cylinder. It may not fill.*

2. Set the regulator flow dial on the cylinder to Off (FIGURE 4.2).
 3. Remove the cylinder and connector fillport covers.
-

CAUTION

DO NOT connect the outlet (cannula) connection to the HomeFill II unit as this will result in the malfunction of the conserving cylinder.

-
- To reset the connector momentarily push **DOWN** on the outer ring (sleeve) of the connector fillport until **GREEN** dots are visible (FIGURE 4.1).

NOTE: *If the outer ring (sleeve) is in the UP position (GREEN dots not visible), the connector fillport will be unable to accept the cylinder fillport. Pushing DOWN momentarily will reset the connector fillport (GREEN dots visible) to accept the cylinder fillport.*


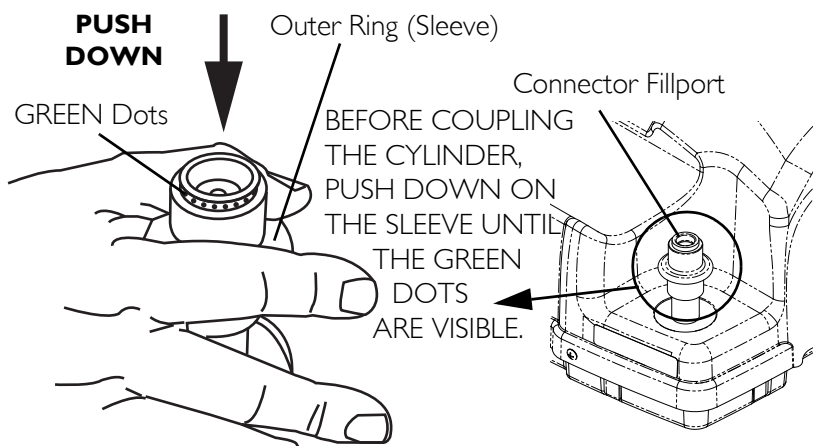
FIGURE 4.1 Resetting Connector Fillport

- Grasp the cylinder/regulator assembly in the area behind the cylinder gauge (FIGURE 4.2).
- Position the cylinder in the compressor cradle (FIGURE 4.2).
- Align the cylinder fillport with the connector fillport (FIGURE 4.2).
- Pull UP on the outer ring (sleeve) of the connector fillport while pushing DOWN on the cylinder/regulator assembly to couple the cylinder fillport into the connector fillport (FIGURE 4.2).

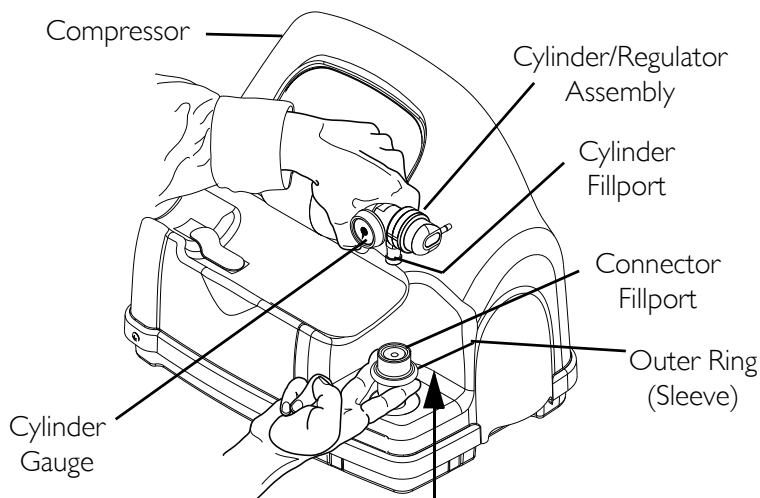NOTE: *An audible “click” indicates that the cylinder is properly connected.*

PULL UP

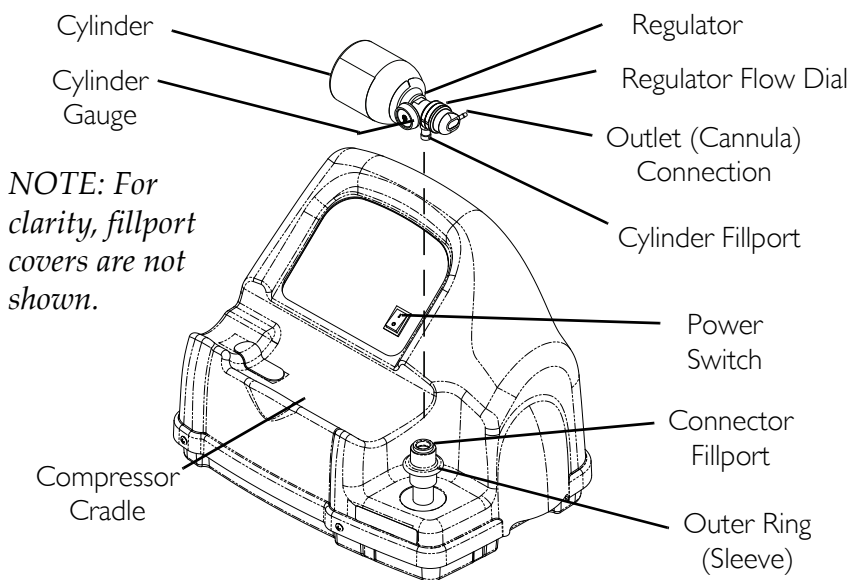
FIGURE 4.2 Connecting the Cylinder to the Compressor

Disconnecting the Cylinder from the Compressor

⚠ WARNING

NEVER use any type of tool to connect/disconnect the cylinder and the compressor. Otherwise, severe injury and/or damage may occur.

NOTE: For this procedure, refer to FIGURE 4.3 and FIGURE 4.4 on page 26.

1. Press the compressor power switch to the Off (O) position.
2. Grasp the cylinder/regulator assembly in the area behind the cylinder gauge.
3. With the other hand, grasp the outer ring (sleeve) of the connector fillport and push DOWN.
4. Lift up on the cylinder/regulator assembly to remove from the connector fillport.

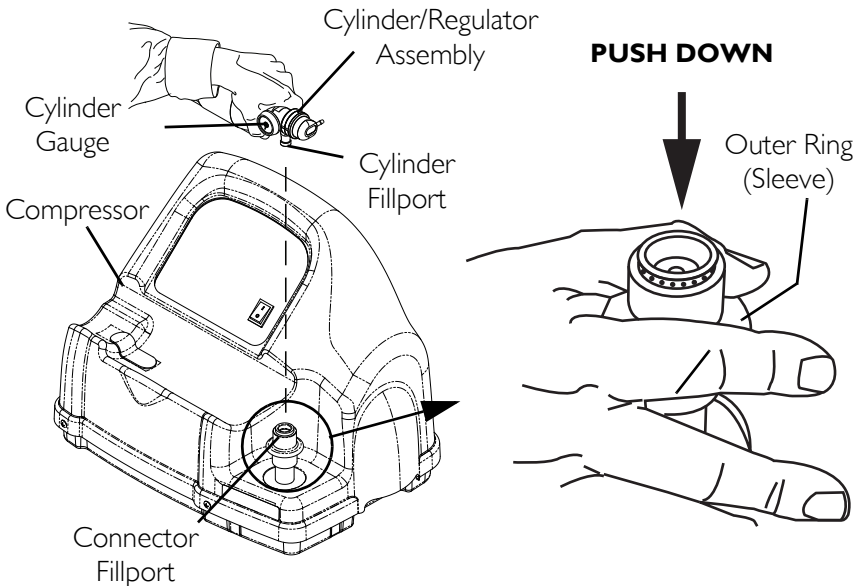
FIGURE 4.3 Disconnecting the Cylinder from the Compressor

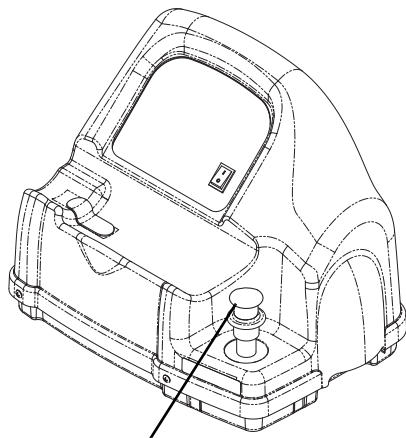
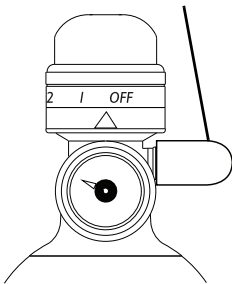
-
- When the cylinder fillport is disconnected from the connector fillport, release the outer ring (sleeve) of the connector fillport and use both hands to remove the cylinder from the compressor cradle.

⚠ WARNING

The fillport cover on the connector fillport and the cylinder fillport **MUST be replaced after filling and when not in use. If either of the fillport covers are missing contact your healthcare provider before using.**

- Place the fillport covers onto the connector fillport and cylinder fillport.
- Examine the cylinder gauge to ensure the cylinder is full (needle pointing into GREEN area). If the cylinder is not full, perform the following:
 - Repeat Connecting the Cylinder to the Compressor on page 22.
 - If the cylinder is not full after a second attempt to fill it, contact the provider.

Cylinder Fillport Cover

Connector Fillport Cover

FIGURE 4.4 Fillport Covers

Turning the Compressor On

NOTE: For this procedure, refer to FIGURE 4.5 on page 28.

1. Make sure the concentrator is On. Refer to the concentrator Owner's Manual.

CAUTION

The concentrator flow rate to the patient **MUST be set at a maximum of 2.5 L/min. or less for Platinum 5 models and 5 L/min. for Platinum 9 models. Otherwise, the oxygen level to the compressor will be of <90% and the compressor will not fill the cylinder.**

-
2. Set the concentrator flow rate to the patient to 2.5 L/min. or less for Platinum 5 and 5 L/min. for Platinum 9 oxygen concentrator. Refer to the concentrator Owner's Manual.
 3. Make sure the cylinder is connected to the compressor. Refer to Connecting the Cylinder to the Compressor on page 22.
 4. Push the power switch on the control panel to the On (I) position.
 5. Examine the control panel. The following sequence of events should occur:
 - A. 0-3 minutes - The O₂ below normal (YELLOW) light will light up while the compressor warms up.
 - B. After three minutes - The filling (GREEN) light will be on while the cylinder is filling.
 - C. The full (GREEN) light will be on when the cylinder is finished filling. Proceed to step 6.

NOTE: The O₂ below normal (YELLOW) light may come on if the oxygen level from the concentrator has not yet reached or drops below 90%. The compressor will begin or resume filling when the

oxygen level reaches 90%. Refer to Troubleshooting/ Cylinder Fill Times on page 33 if the light does not go Off within 10 minutes.

NOTE: Refer to Troubleshooting/ Cylinder Fill Times on page 33 if the Attention (red) light and audible alarm are on.

6. Remove the full cylinder. Refer to Disconnecting the Cylinder from the Compressor on page 25.
7. Push the power switch to the Off (O) position.

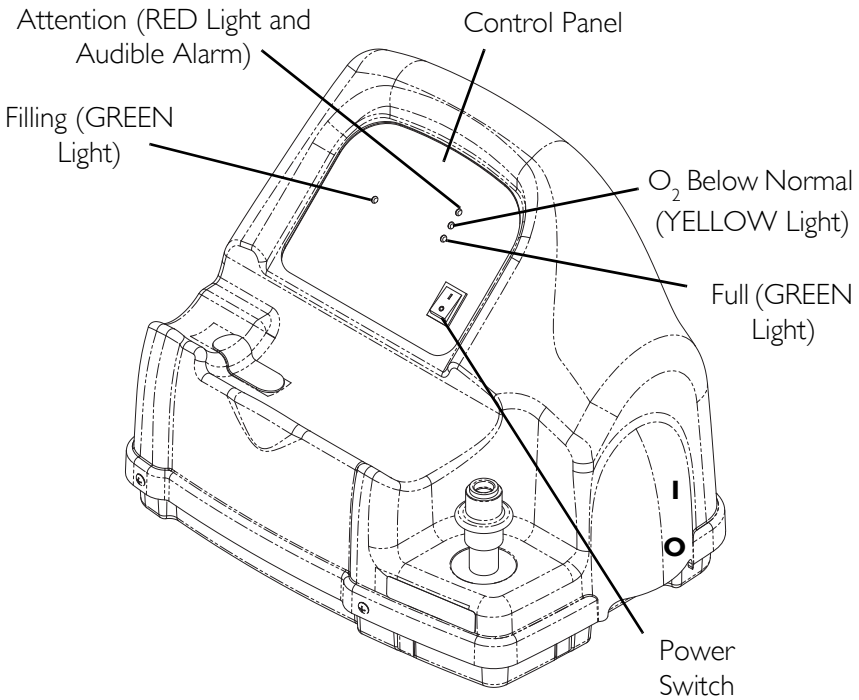
FIGURE 4.5 Turning the Compressor On

Setting the Flow Knob to the Prescribed Setting

NOTE: For this procedure, refer to FIGURE 4.6.

1. Attach the nasal cannula to the outlet fitting of the cylinder.

-
- Turn the flow knob to the L/min. setting prescribed by your doctor or therapist.
-

⚠ WARNING

As changing the L/min. setting on the flow knob will affect the dose of oxygen delivered, DO NOT readjust the L/min. setting unless your doctor or therapist instructs you to.

- To turn the flow of oxygen Off, turn the flow knob counter-clockwise to the Off position.

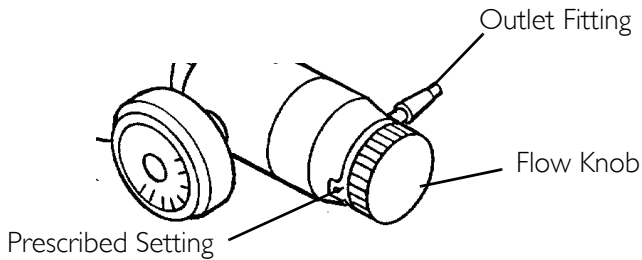
FIGURE 4.6 Setting the Flow Knob to the Prescribed Setting

Technical Description

The HomeFill II Oxygen System (IOH200AW) is an electro-mechanical prescription system for home use. Its intended function and use is to supply pressurized oxygen to fill Invacare proprietary gas cylinders for ambulatory use while the patient receives therapeutic oxygen from an accessory oxygen concentrator. The IOH200AW gas system is designed for use with an Invacare Platinum oxygen concentrator with an integral HF2 accessory port that can supply a gas concentration greater than 90% O₂ around 2 L/min. while supplying a patient therapy flow rate up to 2.5 L/min. for Platinum 5 and 5 L/min. for Platinum 9.

The HomeFill II Oxygen system consists of an IOH200AW gas compressor, connection tubing, and the Invacare supplied portable gas cylinders with the integral flow regulator and interlocking fill fitting.

The IOH200AW compressor consists of a multistage compressor, internal tank, control electronics with an integral oxygen concentration sensor, interlocking fill fitting, inlet fitting, and drive motor. Compressor operation is controlled by the electronics assembly. The motor/compressor is allowed to fill a cylinder only when the oxygen concentration of the inlet gas stream is greater than 90%. The motor/compressor fill operation is completed when the gas in a cylinder reaches the maximum fill pressure of 2000 PSI (13786 kPa).

The IOH200AW compressor can fill cylinders for portable use by ambulatory patients. Invacare supplied portable gas cylinders have an interlocking fill fitting that matches only with the Invacare IOH200AW compressor assembly. The gas cylinder also contains an integral flow regulator with fixed, selectable flow settings.

SECTION 5—INDICATOR LIGHTS

INDICATOR LIGHT COLOUR	COMPRESSOR STATUS	EFFECT
None	Compressor is Off.	O ₂ flowing to user if concentrator is On.
O ₂ Below Normal - YELLOW	Compressor is On and warming up (0-3 minutes after power is On).	Cylinder filling not started.
		O ₂ flowing to user if concentrator is On.
	O ₂ from concentrator is <90% (anytime after three minute warmup).	Cylinder filling stops, will restart when O ₂ >90%.
		O ₂ flowing to user if concentrator is On.
Filling - GREEN	Compressor is On and cylinder is filling. O ₂ from concentrator is >90% (any time after three minute warmup).	Cylinder is filling
		O ₂ flowing to user if concentrator is On.
Full - GREEN	Compressor is On. Cylinder has finished filling (full) -remove cylinder (FULL light is On in >10 mins.) or Cylinder is not attached properly - check connection (FULL light is On in <10 mins.).	Cylinder filling stops Cannot fill another cylinder until compressor is turned Off and On again O ₂ flowing to user if concentrator is On.

INDICATOR LIGHT COLOUR	COMPRESSOR STATUS	EFFECT
ATTENTION - RED (Audible Alarm)	Compressor is On. Cylinder NOT filling Causes: Full cylinder attached when unit turned On - replace with empty cylinder or Internal failure - contact supplier.	Compressor does not start filling or stops filling Unit MUST be turned Off and On before filling can occur. O ₂ flowing to user if concentrator is On.

SECTION 6—TROUBLESHOOTING/ CYLINDER FILL TIMES

TROUBLESHOOTING	
PROBLEM	SOLUTION
There are no indicator lights on when compressor is turned on.	<ol style="list-style-type: none"> 1. Turn compressor Off (O). 2. Check that the power cord is correctly plugged into compressor and wall socket. 3. Turn the compressor on (I). 4. Clean air filters. Refer to <u>Maintenance</u> on page 36. 5. If performing STEPS 1-4 does not cause any indicator lights to come on, contact your home health care provider or Invacare for service.
RED light On	<ol style="list-style-type: none"> 1. Check the cylinder gauge to see if it is full. Cylinder full - Remove the cylinder and turn the unit Off (O). Perform the <u>Cylinder Prefill Inspection</u> on page 19 with a cylinder that is not full. Cylinder not full - Proceed to STEP 2. 2. Check that the cylinder is properly connected. Turn the compressor Off (O). Remove the cylinder and reinstall. Turn the compressor On (I). 3. If the cylinder is not full and is properly connected, the compressor may have an internal failure. Contact your home health care provider or Invacare.
Compressor cycles On and Off or O ₂ Below Normal - alternating (YELLOW) and Filling (GREEN) lights or O ₂ Below Normal (YELLOW) light goes On and Off	<ol style="list-style-type: none"> 1. Check the concentrator liter flow setting is at 2.5 L/min. or less. Where necessary, adjust the litre flow setting (refer to the concentrator owner's manual). Wait at least 20 minutes for the GREEN light to appear. If it does not appear, proceed to STEP 2. 2. The concentrator may need service. Contact your home health care provider or Invacare.

TROUBLESHOOTING

PROBLEM	SOLUTION
<p>O₂ Below Normal (YELLOW) light stays On.</p>	<ol style="list-style-type: none"> 1. Ensure the concentrator has warmed up for at least 20 minutes. If 20 minutes have not passed, turn the compressor Off (O) until the concentrator warms up. If the YELLOW light remains on proceed to STEP 2. 2. Inspect the connection between the compressor and the concentrator for damage or pinching. If damaged, replace. If pinched, straighten tubing. Turn the compressor Off (O) for at least 10 seconds. Turn the compressor On (I). If the YELLOW light remains on after 10 minutes, proceed to STEP 3. 3. The concentrator may need service. Contact your home health care provider or Invacare.
<p>Fill times exceed maximum recommended fill times or GREEN filling light stays On.</p>	<ol style="list-style-type: none"> 1. Ensure the cylinder regulator is set to Off. 2. Check the recommended fill times shown in <u>Cylinder Fill Times</u> on page 35. 3. Check that the cylinder is properly connected. Turn the compressor Off (O). Remove the cylinder and reinstall. Turn the compressor ON (I). 4. Check the compressor/concentrator line for proper connection and/or damage (i.e. - leaks or kinks). 5. Check the concentrator litre flow setting is at three L/min. or less. Where necessary, adjust the litre flow setting (refer to the concentrator owner's manual). If the GREEN filling light does not go Off, the compressor and/or concentrator may need service. Contact your home health care provider or Invacare.
<p>Excessive noise when compressor is On.</p>	<ol style="list-style-type: none"> 1. Contact your home health care provider or Invacare for service.
<p>Hissing sound from Regulator when flow selector is Off.</p>	<ol style="list-style-type: none"> 1. Turn flow selector Off. 2. Should hissing sound continue, contact your home health care provider or Invacare.

TROUBLESHOOTING

PROBLEM	SOLUTION
Regulator is On but O ₂ is not being delivered.	<ol style="list-style-type: none"> 1. Refill cylinder 2. Check that the cannula is straight and is not pinched. If damaged, replace. 3. Contact your home health care provider or Invacare for service.

CYLINDER FILL TIMES

Cylinder Filling Times	Concentrator Flow Rate to Patient: (Platinum 5 up to 2.5 L/min. and Platinum 9 up to 5 L/min.)			
	*HF2PC9	*HF2PCL4	*HF2PCL6	HF2PCE8
NORMAL	2 hr 20 min	60 min	1 hr 25 min	2 hr 5 min
MAXIMUM	2 hr 45 min	1 hr 20 min	1 hr 50 min	N/A

NOTE: All filling times are approximate and may vary according to environmental conditions.

**NOTE: Not for EU countries.*

SECTION 7—MAINTENANCE

Cleaning the Compressor Filter

⚠ WARNING

Push the power switch to the **Off (O)** position and unplug the compressor before cleaning.

DO NOT operate the compressor when the filter is not installed.

CAUTION

DO NOT unplug by pulling the cord, as this could result in damage.

NOTE: For this procedure, please refer to FIGURE 7.1.

NOTE: All compressors have one filter at the back of the cabinet.

NOTE: This procedures should be performed at least once a week.

1. Unplug the compressor. Hold the plug not the cord.
2. Remove the filter.
3. Clean the filter with a vacuum cleaner or wash in warm soapy water and rinse thoroughly.
4. Dry the filter thoroughly before reinstallation.

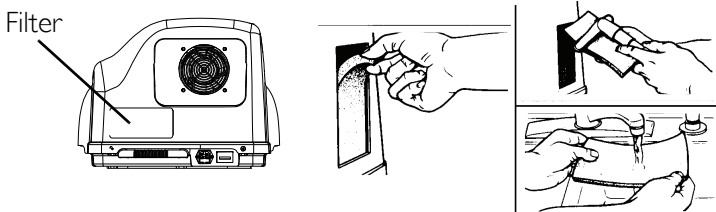
FIGURE 7.1 Cleaning the Compressor Filter

Cabinet

WARNING

Push the power switch to the Off (O) position and unplug the compressor before cleaning. To avoid electric shock, DO NOT remove cabinet.

1. Clean the cabinet with a mild household cleaner and a non-abrasive cloth or sponge.

CUSTOMER SERVICE AND WARRANTY INFORMATION

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Belgium & Luxembourg:

Invacare nv, Autobaan 22, B-8210 Loppem • Tel: (32) (0) 50 83 10 10 •
Fax: (32) (0) 50 83 10 11 • belgium@invacare.com • www.invacare.be

Danmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby • Tel: (45) (0)36 90 00 00 •
Fax: (45) (0)36 90 00 01 • denmark@invacare.com • www.invacare.dk

Deutschland:

Invacare GmbH, Alemannenstraße 10, D-88316 Isny • Tel: (49) (0)75 62 7 00 0 •
Fax: (49) (0)75 62 7 00 66 • kontakt@invacare.com • www.invacare.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica • Tel: (49) (0)57 31 754 540 •
Fax: (49) (0)57 31 754 541 • edo@invacare.com • www.invacare.eu.com

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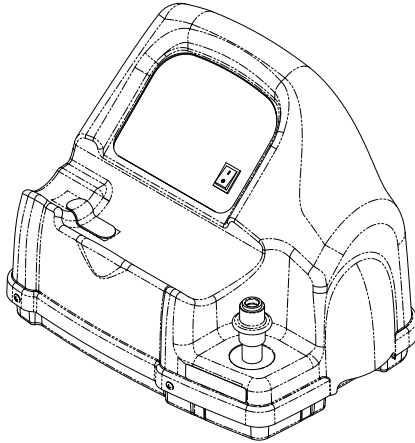
Making Life's Experiences Possible™

Invacare Corporation

www.invacare.com

Yes, you can.

USA

One Invacare Way
Elyria, Ohio USA
44036-2125
440-329-6000
800-333-6900
Technical Services
440-329-6593
800-832-4707

Manufacturer:
Invacare Rehabilitation
Equipment (Suzhou) Co., Ltd.
No. 435 Xieyu Street, SIP,
Suzhou, Jiangsu, PRC
215026

Tel: 86-512-62586180
Fax: 86-512-62586167

EU Representative
Invacare Deutschland
GmbH
Kleiststraße 49, D-32457
Porta Westfalica
Tel: +49 (0) 5731 754 0
Fax: +49 (0) 5731 754 52191

Part No 1145805

Rev D - 07/12

© 2012 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by™ and®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

3-in-1 Oil is a registered trademark of American Home Products Corporation. WD-40 is a registered trademark of the WD-40 Company.