Porto NG

Douro

Faro

Fino

Cosy UP

Move UP

Invacare® Porto NG
Invacare® Douro
Invacare® Faro
Invacare® Fino
Invacare® Cosy UP
Invacare® Move UP

en	Lift-Out Chair User Manual	3
es	Sillón elevador Manual del usuario	21
fr	Fauteuil releveur Manuel d'utilisation.....	41
it	Poltrona elevabile Manuale d'uso.....	61
pt	Poltrona Manual de utilização	81

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for
future reference.

Yes, you can.®

©2023 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Contents

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

1 General	4
1.1 Introduction	4
1.2 Symbols in this Document	4
1.3 Intended Use	5
1.4 Limitation of Liability	5
1.5 Warranty Information	5
2 Safety	6
2.1 General Safety Information	6
3 Setup	8
3.1 Components	8
3.2 Unpacking	8
3.3 Installation	9
3.3.1 Assembly of motor-driven models	9
3.3.2 Disassembly of motor-driven models	9
3.3.3 Assembly of manually operated models	10
3.3.4 Disassembly of manually operated models	10
3.3.5 Assembly Cosy UP	11
3.3.6 Removing the Cosy UP cushion	11
4 Usage	14
4.1 Operating the Lift-Out Chair	14
4.1.1 Activating / Deactivating the four-wheel system	15
5 Maintenance	16
5.1 Maintenance instructions	16
5.2 Replacement of components	16
5.3 Cleaning and disinfection	16
5.4 Troubleshooting	17
5.5 Waste disposal	17

6 Technical Data	18
6.1 Technical characteristics	18

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Only use this product if you have read and understood this manual. Seek additional advice from a healthcare professional who is familiar with your medical condition and clarify any questions regarding the correct use and necessary adjustment with the healthcare professional.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

Other Symbols

(Not applicable for all manuals)

UK Responsible Person
Indicates if a product is not manufactured in the UK.

Triman
Indicates recycling and sorting rules (only relevant for France).

1.3 Intended Use

Intended use

The Lift-Out Chair is an armchair with lifter function, designed for indoor use at home. Designed to provide maximum assistance, the Lift-Out Chair helps you get to an upright position both safely and easily.

The Lift-Out Chair is not intended for prolonged use. Four consecutive hours is the maximum recommended by Invacare.

Intended users

The Lift-Out Chair is intended for persons with a maximum user weight, as stated in *6 Technical Data, page 18*.

Indications / contra-indications

This Lift-out chair offers various seating or lying positions and the lifter function allows you to sit down and get up without effort. Changing position regularly improves comfort and vascularisation of the lower limbs.

There are no contra-indications known if the Lift-Out chair is used as intended.

1.4 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

2 Safety

2.1 General Safety Information

WARNING!

Risk of injury or damage

- The Lift-Out Chair is to be used by one person at a time.
- Do not sit on the leg rest or armrests, they are not designed for this purpose.
- Do not lean on the leg or backrest when sitting down, as this could cause the Lift-Out Chair to tip over.
- The Lift-Out Chair **MUST** be used with its seat cushion and backrest.
- Do not try to sit in the armchair when it is in the raised position. Always leave the armchair in the seated position when not in use.
- Always place the Lift-Out Chair on a completely flat surface to ensure optimal stability during use.

WARNING!

Risk of injury

- Before installing or using the armchair, ensure that no person (more particularly a child) or animal is present within the area of operation of the armchair.
- The leg rest folds in as it closes and could injure a child. Keep children's hands and feet well clear of the leg rest and the scissor or elevator mechanism.
- Ensure that arms and legs are not within the perimeter of the armchair's elevator mechanism or in contact with it. If it is necessary to remove objects that have fallen under the armchair, it is imperative to disconnect the power supply beforehand.

DANGER!

Risk of Death or Serious injury

- The cover of the product is not designed to be fire-resistant.
- Do not smoke in the Lift-Out Chair, if you have difficulties getting up or if your mobility is impaired.
 - If any part of the armchair should catch fire, only use an approved powder type fire extinguisher.
 - Never place the armchair close to a major source of heat or ignition (fireplace, electric heater, gas heater, oil heater, etc).
 - Do not expose the Lift-Out Chair to direct sunlight (UV) for a prolonged period of time.

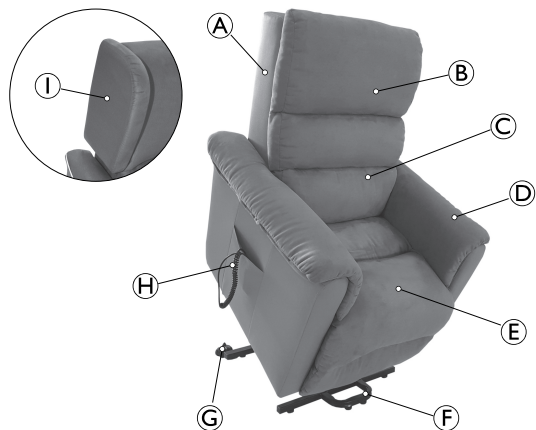
**NOTICE!**

Risk of damage to property

- Use protections of headrest and armrests (available as an option) to protect Lift-out chairs with vinyl cover in case of strong acid sweating.
- The fabric cover is not designed to withstand continuous friction such as spasmodic movements, for example.

3 Setup

3.1 Components

- Ⓐ Backrest
- Ⓑ Removable back cushion (Cosy UP only)
- Ⓒ Lumbar support (Porto NG and Cosy UP only)
- Ⓓ Armrest (Fino and Cosy UP with removable cushion)
- Ⓔ Seat cushion
- Ⓕ Feet (2)
- Ⓖ Castors (2)
- Ⓗ Hand control
- Ⓘ Lateral support (Douro and Porto NG only)

3.2 Unpacking

WARNING!

Risk of injury

- Inspect the Lift-Out Chair to check for any damage, nicks, tears or damage to the frame.
- If you notice any defects, do not attempt to use the Lift-Out Chair. Contact the Invacare provider or carrier for instructions on how to proceed.

NOTICE!

The cardboard packaging protects the Lift-Out Chair during transport.

- Open the cardboard box carefully.
- Do not use sharp objects to open.

1. Open the flaps of the cardboard box.
2. Remove the unassembled backrest.
3. Tear open the cardboard box at the four corners to remove the Lift-Out Chair.
4. Remove the protective plastic film and the anti-humidity sachets.

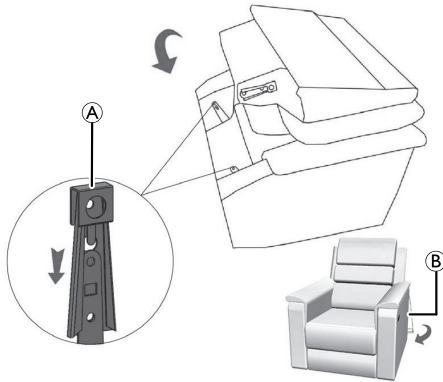
3.3 Installation

WARNING!
Risk of injury

- Leave sufficient room (approximately 15 cm) behind the Lift-Out Chair to prevent it from coming into contact with the wall when in reclined position.
- Always place the Lift-Out Chair on a completely flat surface to ensure optimal stability during use.
- Do not place the Lift-Out Chair near a staircase.

3.3.1 Assembly of motor-driven models


1. Align the backrest with the seat to fit the locking mechanism (A).
2. Slide the backrest down until you hear a clicking sound. The mechanism on both sides should be locked (A).
3. Close the cover (B) behind of the backrest .


Make sure the backrest is properly locked on the seat, by pulling firmly at the backrest. If the backrest doesn't stay fixed, repeat the assembly.

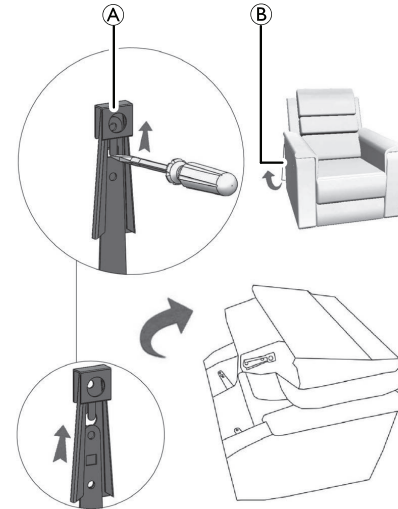3.3.2 Disassembly of motor-driven models

1. Open the cover behind of the backrest ②.
2. Use a screwdriver to release the locking mechanism between the backrest and the seat on the left and the right side ①.
3. Move the backrest forward to disassemble completely.

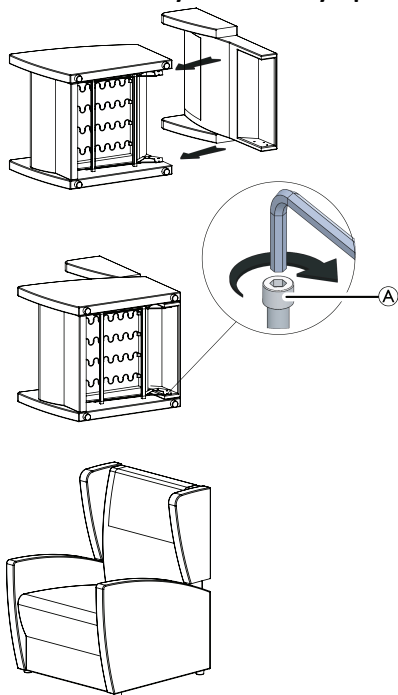3.3.3 Assembly of manually operated models


1. Lay the chair on the side to get better access.
2. Align the brackets of the seat with the holes of the backrest plate on the right and left side.
3. Use an Allen key to tighten the bolts ①.
4. Place the chair upright again.


Make sure the backrest is properly locked on the frame, by pulling firmly at the backrest. If the backrest doesn't stay fixed, repeat the assembly.

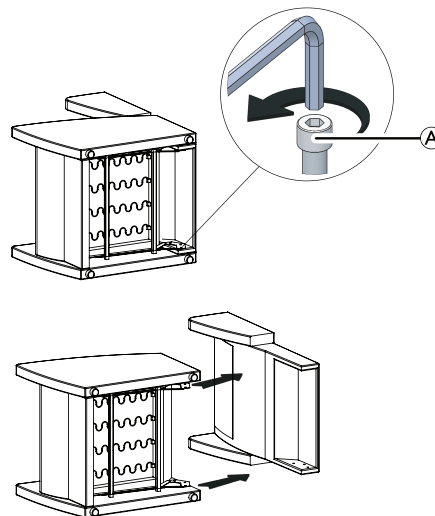3.3.4 Disassembly of manually operated models

1. Lay the chair on the side to get better access.
2. Use an Allen key to untighten the bolts **A**.
3. Slide the backrest to separate from the seat.

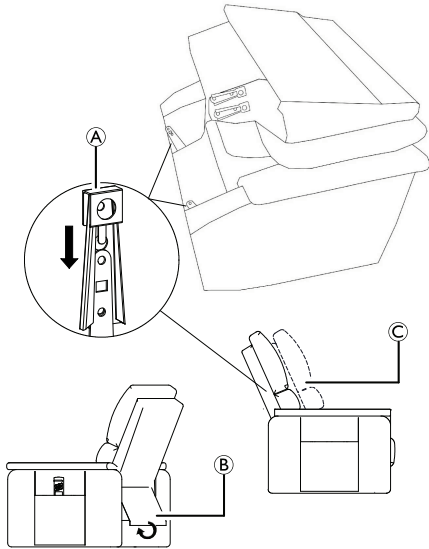3.3.5 Assembly Cosy UP


1. Align the backrest with the seat to fit the locking mechanism **A**.
2. Slide the backrest down until you hear a clicking sound. The mechanism on both sides should be locked **A**.
3. The two brackets on the backrest define two depth positions **C**.
4. Close the cover **B** behind of the backrest .

Make sure the backrest is properly locked on the seat, by pulling firmly at the backrest. If the backrest doesn't stay fixed, repeat the assembly.


3.3.6 Removing the Cosy UP cushion

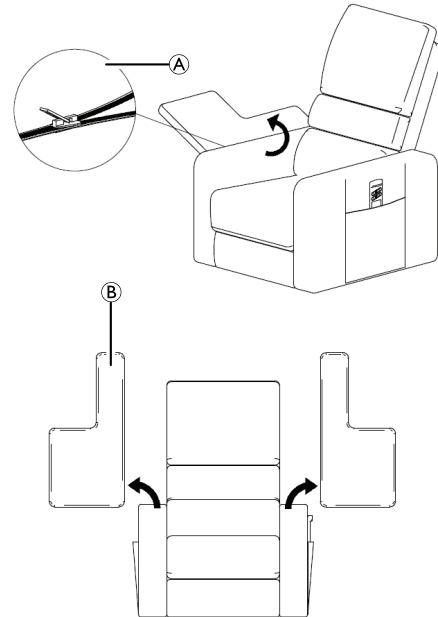- Armrest cushion


1. Raise the flaps on the left and right side and open the zipper **A**.
2. Remove the armrest cushion **B** on both sides.

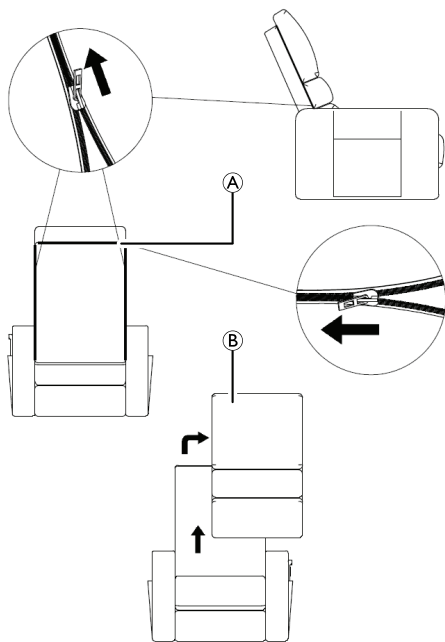
- **Backrest cushion**

1. Open the backrest zipper (A).
2. Remove the backrest cushion (B).

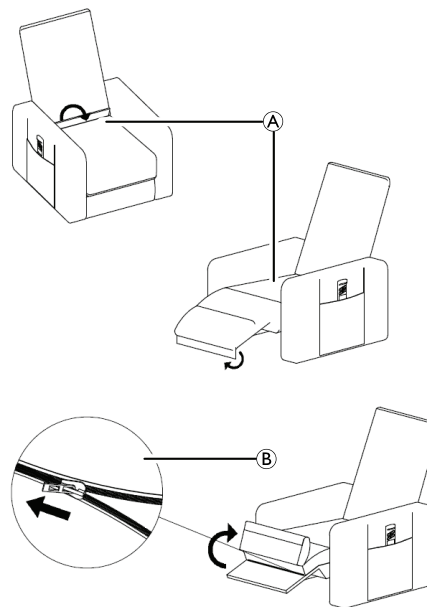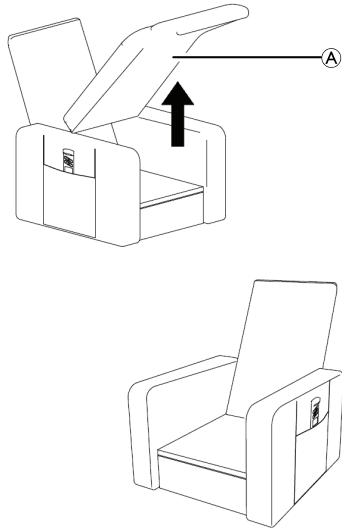- **Seat cushion**

1. Unfold the cushion completely ①.
2. Open the zipper ② between the cushion and the chair.
3. Remove the seat cushion completely ①.

4 Usage

4.1 Operating the Lift-Out Chair

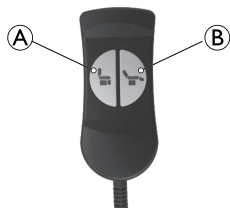
WARNING!
Risk of injury

- Always be sure to centre your weight on the seat cushion regardless of whether or not the armchair is reclined.
- If the motor is used continuously for 2 minutes or more, wait 18 minutes before using it again.
- Do not reposition the armchair when it is in the reclined position.
- Do not reposition the armchair when it is in use or connected to the power supply.
- The remote control is intended to be used by the occupant only.

Manually operated models


1. To come to a reclined position, press against the backrest the armrests.
2. To return to seated position, press the legs on the footrest.

Models with single motor

1. To raise the armchair to the lift-out position, press button **A**.
To return to seated position, press button **B**.
2. To swing up the footrest and lower the backrest to reclined position, press button **B**.
To return to seated position, press button **A**.

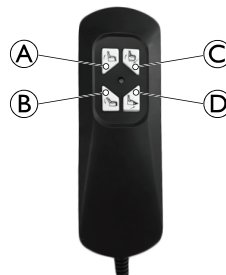Models with two motors (Faro, Porto NG, Cosy UP only)

1. To lower the backrest, press button **B**.
To raise the backrest, press button **A**.
2. To come to the lift-out position, press button **C**.
To return to seated position, press button **D**.
3. To swing up the footrest, press button **D**.
To swing down the footrest, press button **C**.
4. To come to a reclined position, press both buttons **B** and **D**.
To return to seated position, press both buttons **A** and **C**.
5. To raise the backrest and come to the lift-out position, press both buttons **A** and **C**.
To return to seated position, press both buttons **B** and **D**.

4.1.1 Activating / Deactivating the four-wheel system

The **Move UP** chair can be moved. The front wheels must be activated to enable this function.

Activation

- Push down the foot lever, located on the back side of the chair.

Deactivation

This locks the chair in place

- Lift the foot lever, located on the back side of the chair.

WARNING!

Risk of injury

To avoid injury when operating the product.

- The user must be properly seated and secured before activating the pedal.
- The chair must not be used for user transport, only small manoeuvres, and must not be used in unlevelled floor, floors with carpets, etc
- Be sure to deactivate the pedal before using the lift out function.

5 Maintenance

5.1 Maintenance instructions

WARNING!

- Adjustments or repair must only be performed by a qualified technician.
- Disconnect the power supply before maintaining or cleaning the armchair.

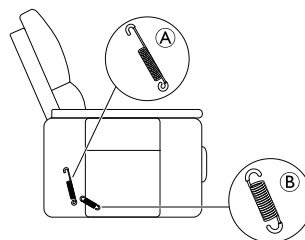
- ❑ Check the armchair for any squeaks or suspect noise from the frame. Lubricate linkage points on the frame, if required.
- ❑ Check the armchair for any defects. If you detect any defects, contact your Invacare provider for servicing.

5.2 Replacement of components


The chair's spring components might need to be replaced, due to natural wear and tear.

-  Please request suitable spare parts from your local Invacare® provider.

Models with single motor

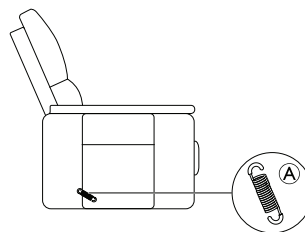Models with two motors (Faro, Porto NG, Cosy UP only)

5.3 Cleaning and disinfection

Cleaning agents must not contain acid, solvent or salt.

- ❑ Regularly clean mechanical components to limit dust build-up in the frame.
- ❑ Clean fabric with fabric cleaning products only. Do not saturate the fabric.
Brush if necessary to restore the original appearance.

- ❑ Clean the hand control with a slightly damp cloth. Do not use a spray.
Dry the hand control thoroughly before using it again.
- ❑ Wash removable cushions in a washing machine at 30°. (Removable armrest cushions on Fino and Cosy UP only. Removable seat and back cushion on Cosy UP only.)

5.4 Troubleshooting

Symptom	Problem	Action
Lift-Out Chair doesn't move to the reclined end positions.	Insufficient space to allow movement of backrest and/ or footrest.	Ensure the Lift-Out Chair is not positioned too close to a wall or an obstacle
No movement	Mains plug not connected	Connect the mains plug
	Hand control not correctly connected	Connect the hand control correctly
		Check that the cable is not damaged
	Mains cable or plug damaged	Contact your Invacare provider for service
Electrical components not correctly connected	Check the connections to the electrical components.	

CAUTION!

Risk of injury

- If none of the described actions solve the problem, do not use the product and contact your Invacare provider.

5.5 Waste disposal

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Waste disposal must comply with the laws and regulations pertaining waste handling in each country.

6 Technical Data

6.1 Technical characteristics

	Faro	Douro	Porto NG	Fino	Cosy UP	Move UP
Voltage	100/240 V					
Frequency	50/60 Hz					
Power	60 W					
Insulation class	Class II					
Degree of protection	IP44					
Fabric material	Heritage / Nappa				Polyester (microfiber) or PVC (vinyl)	Heritage / Nappa
Test for flammability	ISO 1021-1					
Seat foam density	28 kg/m ³					
Armrest foam density	28 kg/m ³				18 kg/m ³	23 kg/m ³

Models

	Faro	Douro	Porto NG	Fino	Cosy UP	Move UP
Relax (manual)		X				
1 motor	X	X	X	X	X	X
2 motors	X		X		X	

Dimensions and weights

	Faro	Douro	Porto NG	Fino	Cosy UP	Move UP
Max. user weight	125 kg	125 kg	125 kg	125 kg	125 kg	120 kg
Total weight	50 kg	57 kg	52 kg	53 kg	57 kg	65 kg
Seat depth	540 mm	540 mm	540 mm	450 mm	450 mm	480 mm
Seat to floor height	470 mm	470 mm	470 mm	470 mm	480 mm	500 mm
Seat width	530 mm	530 mm	530 mm	450 mm	450 mm	480 mm
Max. backrest recline						
Single motor	120°	120°	120°	120°	120°	120°
Two motors	135°		135°		135°	
Total height	1110 mm	1080 mm	1020 mm	1000 mm	1050 mm	1110 mm
Total width	770 mm	740 mm	790 mm	730 mm	800 mm	770 mm
Total depth (seated position)	880 mm	860 mm	910 mm	780 mm	910 mm	970 mm
Total depth (recline position)						
Single motor	1580 mm	1580 mm	1580 mm	1520 mm	1520 mm	1580 mm
Two motors	1670 mm		1670 mm		1720 mm	
Armrest to floor height	600 mm	640 mm	600 mm	670 mm	700 mm	640 mm

Fabric Material and Color

	Faro	Douro	Porto NG	Fino	Cosy UP	Move UP
Nappa Black	X					
Nappa Dark Brown		X	X			
Nappa Grey Taupe					X	

Invacare® Porto NG, Invacare® Douro, Invacare® Faro, Invacare® Fino, Invacare® Cosy UP, Invacare® Move UP

	Faro	Douro	Porto NG	Fino	Cosy UP	Move UP
Nappa Grey						X
Heritage Camel	X	X		X		X
Heritage Grey			X			
Microfiber Taupe (cushions) and PVC Taupe (base)					X	

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Invacare Poirier SAS
J.J. Louro Pereira S.A