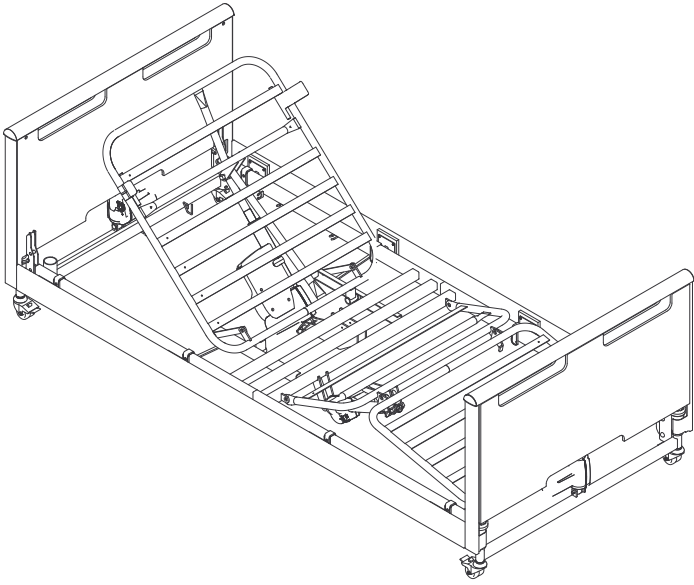
Invacare® Etude® Plus HC

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This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

Yes, you can.®

©2020 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Making Life's Experiences Possible is a registered trademark in the U.S.A.

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Contents

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

1 General	4	7.2 Reconditioning	21
1.1 Introduction	4	8 Troubleshooting	22
1.1.1 Symbols in this Document	4	8.1 Troubleshooting electrical system.	22
1.2 Intended Use	4	9 Technical Data	23
1.2.1 Intended Operator	4	9.1 Dimensions	23
1.3 Service Life	4	9.2 Weights	23
1.4 Compliance	4	9.3 Mattress dimensions	23
1.4.1 Product-specific standards	4	9.4 Environmental conditions	23
1.5 Warranty Information	5	9.5 Electrical system	23
1.6 Limitation of Liability.	5	10 Electromagnetic compatibility (EMC)	24
2 Safety	6	10.1 General EMC information	24
2.1 General safety information	6	10.2 Electromagnetic emission.	24
2.2 Mattresses	7	10.3 Electromagnetic Immunity	24
2.3 Safety information on electromagnetic interference	7	10.3.1 Test specifications for immunity to RF wireless communications equipment	25
2.4 Labels and symbols on the product	7		
2.4.1 Identification Label	7		
2.4.2 Other Labels	8		
3 Setup	10		
3.1 General Safety Information	10		
3.2 Receiving the bed	10		
3.3 Assembly of the bed	10		
3.3.1 Attaching Labels.	11		
3.4 Control Box.	11		
3.5 Wiring	11		
3.6 Dismantle the bed.	11		
4 Operating the bed	12		
4.1 General Safety Information	12		
4.1.1 Low Heights.	12		
4.1.2 Mattress retainers	12		
4.2 Operating the hand control	12		
4.2.1 Locking function.	12		
4.3 Castors and brakes	13		
4.3.1 Castor brake	13		
4.4 Lifting pole	13		
4.4.1 Place the lifting pole	13		
4.4.2 Adjusting the handle height	13		
4.5 Adjusting the leg section	13		
4.6 Emergency positions	13		
4.6.1 Emergency release of a mattress support section	14		
4.7 Mattress support extension	14		
4.8 Transportation and Storage	14		
5 Accessories	15		
5.1 List of available accessories	15		
5.2 Side Rails	15		
5.2.1 Installing side rails	15		
5.2.2 Operating side rails	15		
5.2.3 Verso HC side rail.	15		
5.3 Wooden side panel	16		
5.4 Support handles / Swivel handles	16		
5.5 Transport brackets.	17		
6 Maintenance	19		
6.1 General maintenance information	19		
6.2 Daily inspections	19		
6.3 General servicing information	19		
6.3.1 Checklist maintenance	19		
6.4 Inspection after relocation - Prepare for new user	19		
6.4.1 Checklist - After relocation	19		
6.5 Cleaning and Disinfection	20		
6.5.1 General safety information	20		
6.5.2 Cleaning Intervals.	20		
6.5.3 Cleaning Instructions	20		
6.5.4 Disinfection Instructions	20		
6.6 Lubrication	20		
7 After Use	21		
7.1 Disposal	21		

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.1.1 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

1.2 Intended Use

The bed has been developed for domestic care and long term care:

- *Application environment 3*; Long-term care in a medical area where medical supervision is required and monitoring is provided if necessary and medical electrical equipment used in medical procedures may be provided to help maintain or improve the condition of the patient.

- *Application environment 4*; Care provided in a domestic area where medical electrical equipment is used to alleviate or compensate for an injury, disability or disease.
- The bed is intended for indoor use only.
- The bed is intended for adult users, having a physical size equal to or more than 146 cm, a weight equal to or more than 40 kg and a body mass index (BMI) equal to or more than 17.
- The bed is not intended for transportation of users. It is mobile within a room with a user in it. Castors are lockable.
- The bed is not intended for psychiatric patients.
- Maximum user weight and safe working load are listed on the product label and in the technical data section in this manual.

WARNING!

Any other or incorrect use could lead to hazardous situations.

Invacare accepts no liability for any use, change or assembly of the product, other than stated in this user manual.

1.2.1 Intended Operator

A healthcare professional or private person who has received proper training is the intended operator of this product.

The hand control can also be operated by the occupant of the bed.

1.3 Service Life

The expected service life of this product is seven years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.4 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

1.4.1 Product-specific standards

The product has been tested and conforms to IEC 60601-2-52 (Particular requirements for the basic safety and essential performance of medical beds) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

2 Safety

2.1 General safety information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section.

WARNING!

Risk of injury or damage

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, Invacare provider or qualified technician before attempting to use this product.
- Do not make any unauthorized alterations or modifications to the product.

WARNING!

Risk of entrapment / suffocation

There's a risk of entrapment / suffocation between mattress support, side rail and bed end or between moving parts and objects placed nearby the bed.

- The bed must not be used by persons under 12 years of age, or by persons with a body size equivalent to an average 12 years old or smaller.
- The bed, in combination with side rails must not be used by persons having a physical size less than 146 cm, a weight less than 40 kg or a body mass index (BMI) less than 17.
- Due to mattress compression, an increased risk may occur over time. Periodically monitor gaps between the bed, mattress and/or side rail. Replace mattress if the gaps may lead to entrapment.

WARNING!

Risk of slipping through the openings

The bed fulfils all requirements regarding maximum distances. However, it is possible that persons with small body dimensions slip through the openings between the side rails or through the opening between the side rail and the mattress support.

- Pay special attention, if the bed is used for the care of persons with small body dimensions.

WARNING!

Risk of injury or damage

Inappropriate handling of cables, incorrect connections and use of unauthorized equipment can cause electrical shock and product failure.

- Do not kink, shear or otherwise damage the cables of the product.
- Make sure that no cables are jammed or damaged, when the product is used.
- Disconnect the plug from the mains before moving the bed.
- Make sure the wiring is correct and connections are proper.
- Do not use unauthorized equipment.

WARNING!

Risk of tripping, entanglement or strangulation

Improper routing of cables may cause tripping, entanglement or strangulation hazard.

- Ensure all cables are routed and secured properly.
- Ensure there are no loops of excess cable extending away from the product.

WARNING!

Risk of injury or damage

Ignition sources can cause burns or fire.

- Bed must be placed with a safety space from possible ignition sources (heater, fireplace, etc.)
- The patient and assistants must not smoke while using or operating the bed.

WARNING!

Risk of injury or damage

To avoid injury or damage when operating the product:

- Close supervision is necessary when the product is used near children or pets.
- Do not let children play with the product.

CAUTION!

– For a person entering or exiting the bed, always lower the bed to an appropriate height. The backrest can be used as a support. Make sure, the thigh- and leg section is horizontal to avoid overloading the mattress support.

- Always lower the bed to the lowest position before leaving a person in the bed unattended.
- Make sure that there is nothing under, over, or near the bed that can obstruct the height adjustment, like for example furniture, lifts or window frames.

CAUTION!

Non-original or wrong accessories may affect the function and safety of the product.

- Only use original accessories for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available accessories or contact your local Invacare representative. See addresses at the end of this document.

**CAUTION!****Risk of injury or damage**

The product can get hot when exposed to sunlight or other heat sources.

- Do not expose the product to direct sunlight for prolonged periods.
- Keep the product away from heat sources.

**CAUTION!**

There is a risk of entrapment of fingers in the bed moving parts.

- Pay attention to your fingers.

**IMPORTANT!**

The bed has no isolator (main switch). If the bed needs to be electrically disconnected, it has to be unplugged from the mains socket.

- Always place the bed in a way so the mains socket is easily accessible to unplug the bed.

**IMPORTANT!**

Accumulation of lint, dust and other dirt can impair the product.

- Keep the product clean.

2.2 Mattresses

**WARNING!****Safety aspects regarding combination of side rails and mattresses:**

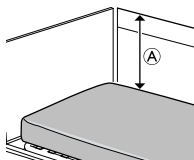
To get the highest possible safety level, when using side rails on the bed, the minimum and maximum measures for mattresses, must be respected.

- For correct mattress measures see mattress table in chapter 9 *Technical Data*, page 23.

**WARNING!****Risk of entrapment and/or suffocation**

– The user could get trapped and/or suffocate, if the horizontal space, between the mattress side and the inside of the side rail, is too big. Follow the minimum width (and length) of mattresses in combination with a side rail, as stated in the mattress table in chapter 9 *Technical Data*, page 23 .

- Be aware that using very thick or soft mattresses (low density), or a combination of these, increases the risk.

**WARNING!****Risk of falling**

The user can fall over the edge and get seriously injured, if the vertical distance \textcircled{A} between the top of the mattress and the edge of the side rail/bed end, is too short. See image above.

- Always keep a minimum distance \textcircled{A} of 22 cm.

- Follow the maximum mattress height in combination with the side rail as stated in the mattress table in chapter 9 *Technical Data*, page 23.

2.3 Safety information on electromagnetic interference

**WARNING!****Risk of malfunction due to electromagnetic interference**

Electromagnetic interference between this product and other electrical equipment can occur and disturb the electrical adjustment functions of this product. To prevent, reduce or eliminate such electromagnetic interference:

- Only use original cables, accessories and spare parts, to not increase electromagnetic emission or reduce electromagnetic immunity of this product.
- Do not use portable radio frequency (RF) communications equipment closer than 30 cm to any part of this product (including cables).
- Do not use this product near active high-frequency surgical equipment and the RF shielded room of a system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.
- If disturbances occur, increase the distance between this product and the other equipment or switch it off.
- Refer to the detailed information and follow the guidance in chapter 10 *Electromagnetic compatibility (EMC)*, page 24.

**WARNING!****Risk of malfunction due to electromagnetic interference**

Do not use this bed adjacent to or stacked with other electrical equipment, than specified in the following, as it could result in improper operation. If such use is necessary, the bed and the other equipment must be closely observed to verify that they are operating normally.

This bed can be used together with Invacare approved accessories and medical electrical equipment connected to the heart (intracardially) or blood vessels (intravascularly) provided that following points are respected:

- Medical electrical equipment should not be fixed on the bed's metallic accessories such as side rails, lifting pole, drip rod, bed ends, etc.
- The medical electrical equipment power supply cord should be kept clear of the accessories or any other moving parts of the bed.

2.4 Labels and symbols on the product

2.4.1 Identification Label

The identification label is placed on the frame of the bed and contains the main product information, including technical data.

SN	Serial number
REF	Reference number
	Manufacturer
	Date of manufacture
MD	Medical device
	Max. user weight
	Max. safe working load
	CLASS II equipment
	Type B applied part
	WEEE conform
CE	European Conformity

Abbreviations for technical data:

- Iin = Incoming Current • AC = Alternating Current
- Uin = Incoming Voltage • Max = maximum
- Int. = Intermittence • min = minutes

For more information about technical data, refer to 9 *Technical Data*, page 23.

2.4.2 Other Labels

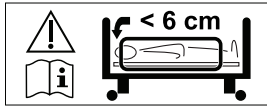
<p>Read the user manual before using this product and follow all instructions for safety and use.</p>

	<p>Definition of min. weight, min. height and min. body mass index of an adult user.</p> <p>See 1.2 <i>Intended Use</i>, page 4 .</p>
	<p>Refer to user documentation for the correct mattress measures.</p> <p>See 9 <i>Technical Data</i>, page 23.</p>

	<p>Total weight of the product with the maximum safe working load applied</p>
	<p>Temperature limit</p>
	<p>Humidity limitation</p>
	<p>Atmospheric pressure limitation</p>
	<p>Transport and storage conditions</p>
	<p>Operation conditions</p>

<p>Indicates areas with increased risk of squeezing feet on the lower sides of the bed ends near the castors.</p> <p>See 4.1.1 <i>Low Heights</i>, page 12</p>	

<p>Indicates detachable parts with a mass of more than 20 kg.</p> <p>See 9 <i>Technical Data</i>, page 23.</p>	

Label on 3/4 length side rails

Indicates the maximum space between side rail and the head bed end.

See the mounting instruction in this manual or the instruction delivered with the side rail.

3 Setup

3.1 General Safety Information

WARNING!

Risk of injury or damage

Damaged parts may affect the safety of the product.

- Check all parts for shipping damage before use.
- In case of damage, do not use the product and contact your Invacare provider for further instructions.

WARNING!

Risk of injury or damage to property

The assembly of the bed and installation of accessories must only be performed by a qualified technician or a person who has received proper training.

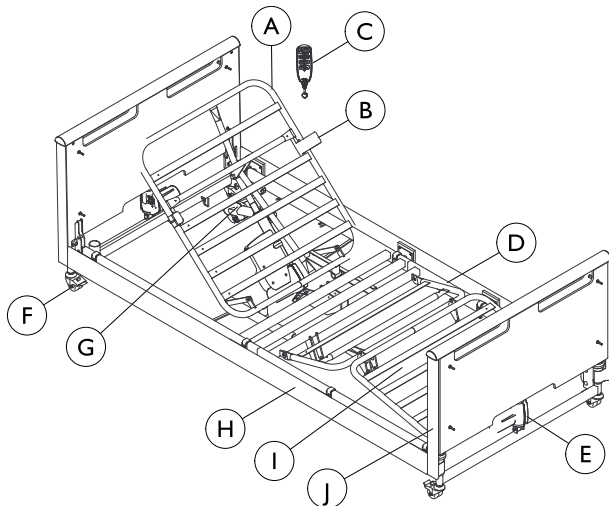
- Follow instructions carefully. If you have any question regarding the assembly, contact your Invacare provider.
- The electrical equipment of the bed must not be dismantled or combined with other electrical equipment.
- After each assembly, check that all fittings are properly tightened and that all parts have the correct function.

IMPORTANT!

- To prevent condensation, the bed should not be used until it has reached operating temperature. See 9 Technical Data, page 23

3.2 Receiving the bed

Scope of delivery:

- (A) Backrest (mattress support, upper half)
- (B) Mattress retainers
- (C) Hand control
- (D) Thigh/Leg section (mattress support, lower half)
- (E) Motor, bed end
- (F) Castors

- (G) Motor, backrest
- (H) Wooden side panel
- (I) Motor, thigh section
- (J) Bed end

Optional

- Support handle
- Swivel support
- Side rail
- Lifting pole
- Mattress support extension
- Back-up battery

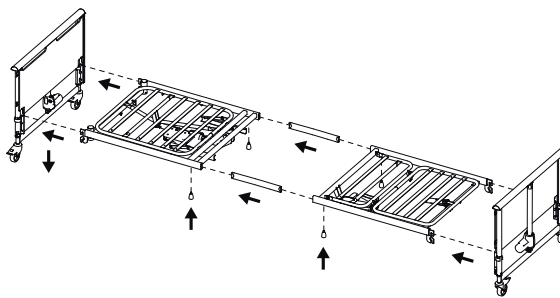
3.3 Assembly of the bed


WARNING!

- After each assembly, check that all fittings are tightened correctly and that the function of all parts is given.

Mattress support

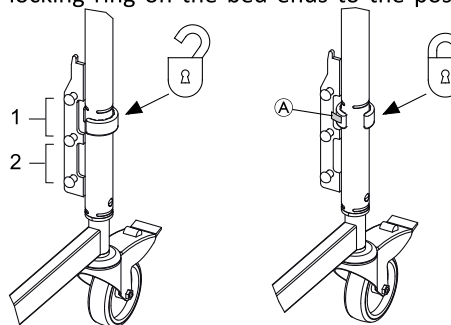
1. Place the two inserts in the upper half of the mattress support. The inserts must be mounted in such a way that one extends further out of the side tube than the other.
2. Loosely screw in the two thumb screws.
3. Push the lower half of the mattress support onto the two inserts.
4. Tighten with two thumb screws.
5. Retighten the two thumb screws at the head end of the top frame.
6. Turn up the four mattress support brackets.
7. Turn the handle of the leg section up.
8. Press the bracket (A) on the locking ring* and turn the locking ring on the bed ends to the position "open".

9. Latch the mattress support to the bed ends and press firmly into position.
10. Make sure that the bed ends are in the same level. It is possible to level the bed in high position (1) and low position (2).
11. Turn the locking ring to the position "lock".

* The locking rings ensure that the mattress support is not accidentally lift clear off the bed ends.

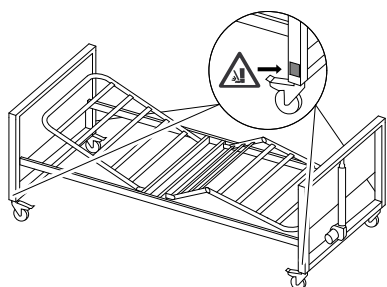
3.3.1 Attaching Labels

1. Place the four foot squeezing warning labels (delivered separately) on the lower sides of the bed ends near the castors.

3.4 Control Box

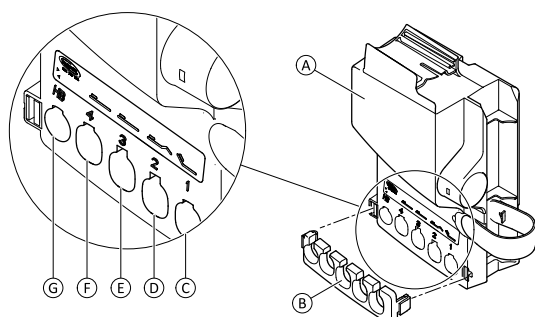The control box is latched to the backrest motor.

The control box (A) is provided with a locking cam (B) and a label with symbols showing where to connect the motor plugs:

- Backrest motor (C)
- Thigh section's motor (D)
- Bed end motor, foot end (E)
- Bed end motor, head end (F)
- Hand control (G)

3.5 Wiring

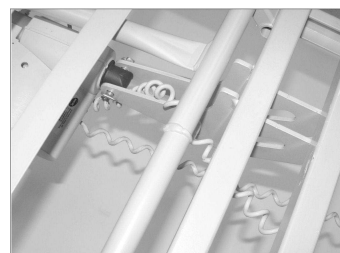
IMPORTANT!

- The cables must be mounted in such a way that they are kept clear off the floor and do not block the castors. Mount the mains cable on the hook for this purpose, see image.


In order to prevent the cables from being damaged when activating the motors, follow the instructions below.


1. Connect the cables of the bed end motors (head and foot end) directly to the control box.
2. The plug of the thigh motor must be directed through the opening of the supporter for the backrest motor and then connected to the control box.

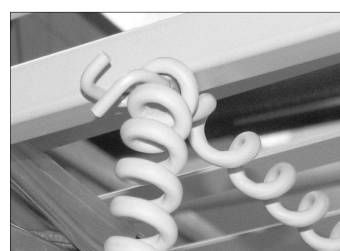3. Connect the main cable plug to the power socket.
4. Run the motors of the bed ends to their top position.
5. Place the motor cable of the foot end on the hook at the foot end.

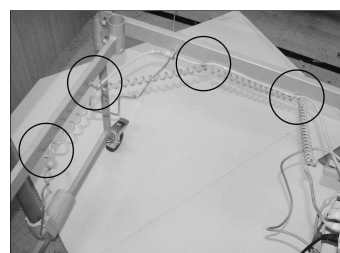6. Run the backrest to its top position.
7. Place the head end motor cables on the four hooks at the head end.

8. Attach the locking cam over the plugs in the control box.

3.6 Dismantle the bed

1. Remove the side rails or wooden side panel and lifting pole.
2. Bring the bed to its lowest position and adjust all mattress support sections to horizontal position.
3. Disconnect the bed from the mains and roll the mains cable onto the hook at the head end of the bed.
4. Remove the locking cam from the control box by means of a tool (e.g. a flat headed screwdriver) and disconnect the cables of the bed end motors and leg section motor.
5. Separate the mattress support from the bed ends.
6. Divide the head and foot end of the mattress support.

4 Operating the bed

4.1 General Safety Information

WARNING!

Risk of personal injury and damage to property.

- The bed must be placed so that the height adjustment is not obstructed by, for example, lifts or furniture.
- Take care that no body parts are being squeezed between fixed parts (such as side rails, bed ends etc) and moving parts.
- The hand control must not be used by children.

WARNING!

Risk of injury

If the bed is used by confused or restless users or users with spasms:

- either, when the bed is equipped with a lockable hand control, lock the hand control functions,
- or make sure that the hand control is out of reach for the user.

4.1.1 Low Heights

CAUTION!

Risk of entrapment or squeezing

- Depending on the configuration, the free space between moving and fixed parts of the bed can become very small when the bed is lowered.
- Ensure your feet are not near or under moving parts when lowering the bed.
 - Ensure cables are routed properly and can not get squeezed when lowering the bed.

CAUTION!

Risk of injury

- Depending on the configuration, the minimum working height of the bed is very low. Using an improper/incorrect posture while working at low working heights can lead to injuries for the carer.
- Be aware of your body posture when attending to the user.

4.1.2 Mattress retainers

WARNING!

Risk of injury

Without properly installed mattress retainers, the mattress can slip sideways and cause the user to fall out of the bed.

- Never use the bed without mattress retainers.
- Always make sure the mattress retainers are properly installed and facing upwards.
- Only use the original mattress retainers.

4.2 Operating the hand control

The hand control can be equipped with three (HB83/HL83) or four (HB84/HL84) buttons to operate the electrical functions of the bed.

Backrest section

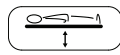
1. Up: press left side of the button (▲).
2. Down: press right side of the button (▼).

Thigh section

1. Up: press left side of the button (▲).
2. Down: press right side of the button (▼).

Height adjustment

1. Up: press left side of the button (▲).
2. Down: press right side of the button (▼).

Tilt function (HB84/HL84)

CAUTION!

Risk of fatal injury

Tilting with head end down can have fatal effect on users who are sensitive for increased blood pressure in the upper part of the body.

- Only medically trained persons must operate the tilt function.
- Always perform a medical evaluation, before tilting the bed with the user's lower extremities positioned higher than the heart.
- The tilt function is NOT a Trendelenburger function and must not be used for medical treatment.

1. Head end up: press left side of the button (▲).
2. Foot end up: press right side of the button (▼).

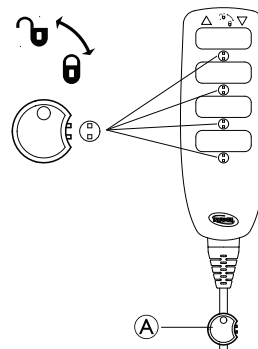
If the bed is equipped with anti-tilt function only, tilting with the foot end up is not possible:

1. Head end up: press left side of the button (▲).
2. Horizontal position: bring the bed either all the way up or all the way down by using the height adjustment button.

4.2.1 Locking function

The locking function prevents usage of certain function buttons. Regardless of how many buttons your hand control has, they can all be locked individually.

1. Insert key Ⓐ in the key hole, below the wanted function.
2. To lock, turn the key clockwise.
3. To unlock, turn the key anti clockwise.

4.3 Castors and brakes

CAUTION!
Risk of trapping/squeezing
 All brakes are foot-operated.
 – Do not release the brake with the fingers.

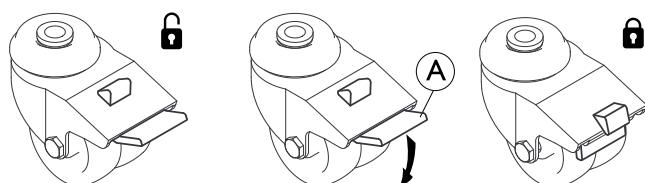
CAUTION!
Risk of injury
 User can fall, while entering or exiting the bed, if brakes are not locked.
 – Always lock the brakes before the user is moving in or out of bed or when attending to the user.
 – At least one castor at the head end and one castor in the foot end must be locked.

i Castors might make marks on different types of absorbing floor surfaces - such as untreated or badly treated floors. To prevent marks, *Invacare*® recommends to place a suitable kind of protection between the castors and the floor.


4.3.1 Castor brake

Locking the brake

Step on the outer pedal (A).

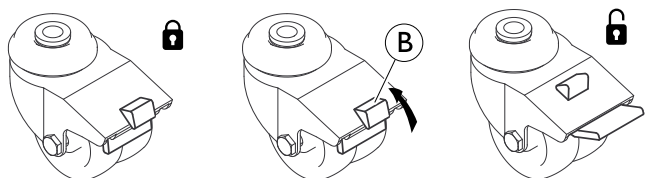Unlocking the brake

Push up the release button (B).

4.4 Lifting pole

WARNING!
Risk of injury
 The bed can tip if the handle is used, while the lifting pole is turned away from the bed.
 – The lifting pole always has to be positioned with the handle hanging over the bed area.
 – Do not exceed the maximum load of the lifting pole; 80 kg.

4.4.1 Place the lifting pole

The lifting pole can be placed either on the left or the right side of the head end of the bed.

1. Insert the lifting pole into the lifting pole tube and fix it with the finger screw.

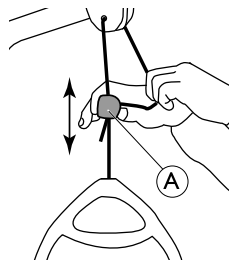
It's not required to tighten the finger screw, in case you want the lifting pole to swing away to the side of the bed.


4.4.2 Adjusting the handle height

The handle height should always be adjusted to the user's need.

1.

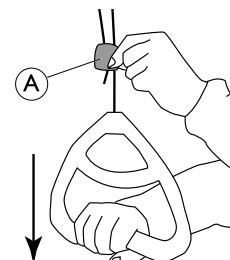2.


1. Hold the cord out while sliding the plastic cord lock (A) upwards or downwards until the handle reaches the preferred height.
2. Lock the handle by pressing back the cord in the lock (A) and pull the handle downwards.

IMPORTANT!
 After adjusting the handle height:
 – Check that the two cords above the cord lock are parallel and inside the cord lock.
 – Make sure the cord is properly locked by pulling the handle hard.

4.5 Adjusting the leg section

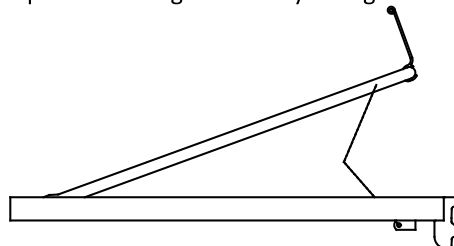Operate the leg section by lifting the mattress handle:

1. **Up:** Lift the mattress handle on the leg section.
2. **Down:** Lift the mattress handle on the leg section right up and then lower the leg section.

4.6 Emergency positions

In case of a medical emergency, it might be required to bring all or certain mattress support sections into a flattened position - e.g. a flattened back section for cardiopulmonary resuscitation (CPR).

To bring a mattress support section into a flattened position,

- use the corresponding function on the hand control

OR

- in case of power failure or urgent need for action, use the manual emergency release and lower the mattress support section.

4.6.1 Emergency release of a mattress support section

In case of a power- or motor failure, an emergency release of the back, thigh or leg section could be necessary. An emergency release of the height adjustment is NOT possible.

CAUTION!
Risk of injury

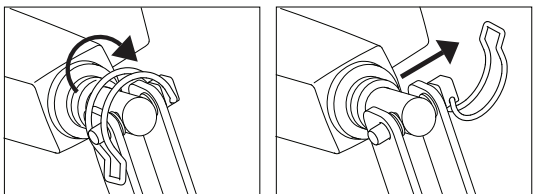
- A minimum of two persons is required for an emergency release of a mattress support section.
- When releasing a mattress support section, it might lower fast. Do not reach under the mattress support while lowering it.

IMPORTANT!

- Before an emergency release of the mattress support, remove the plug from the mains socket.

1. Both persons hold the mattress section.
2. One of them locates the motor in question and pulls out the safety pin.


3. Both persons slowly lower the mattress section until it is completely down.

4.7 Mattress support extension

For users, taller than two meters it is recommended to use a mattress support extension.

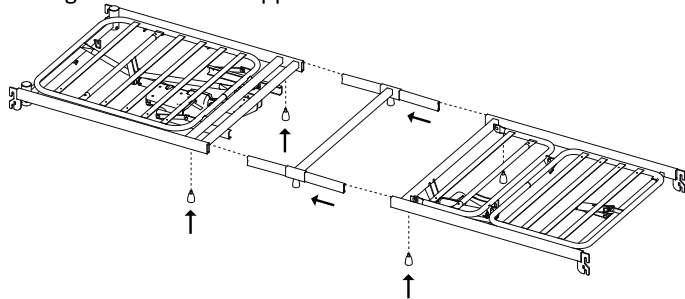Fitting the mattress support extension

1. Disassemble the bed and remove the inserts.
2. Mount the mattress support extension between both parts of the mattress support and reassemble the bed.

4.8 Transportation and Storage

The bed can be transported or stored in assembled or disassembled state.

See 3.6 *Dismantle the bed*, page 11 for detailed instructions on how to disassemble the bed.

To transport or store the bed in its assembled state:

1. Disconnect the power cable from the mains socket.
2. Coil up the power cable and place it onto the hook at the head end of the bed, so it is kept clear of the floor and damage during transport or storage is prevented.

5 Accessories

5.1 List of available accessories

 Due to regional differences, refer to your local Invacare website or catalog for details about available accessories or contact your Invacare provider.

- Verso HC
3/4 length collapsible steel side rail (150 cm x 40 cm), mounted on the side tube. Release button in the foot section
- Wooden side panel
- Mattress support extension – 10 cm or 20 cm
- Fixed support handles – 40 cm x 30 cm or 40 cm x 40 cm
- Swiveling support handles
- Lifting pole
- Transport fitting
- Backup battery

! **Important**
– Use only original accessories and spare parts. Spare parts lists and extra user manuals can be ordered from Invacare.

5.2 Side Rails

5.2.1 Installing side rails

! **WARNING!**
Risk of personal damage
There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.
– Pay attention to your fingers.
– Follow instructions carefully.
– After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

Removable 3/4 length side rails

! **WARNING!**
Risk of entrapment
There's a risk of entrapment or suffocation between mattress support, side rail and bed end.
– When using removable side rails, always make sure that the distance between the bed end and the side rail's handle upper edge is less than 6 cm in the head end and more than 32 cm in the foot end of the bed.

5.2.2 Operating side rails

! **WARNING!**
Risk of entrapment or suffocation
There's a risk of entrapment or suffocation between mattress support, side rail and bed end.
– Always ensure correct fitting and tightening of the side rails.

! **WARNING!**
Risk of falling
If it has been decided that the user needs side rails:
– Never leave the user unattended in the bed with the side rail down.
– Make sure the side rail is in its highest position and properly locked while leaving the user unattended.

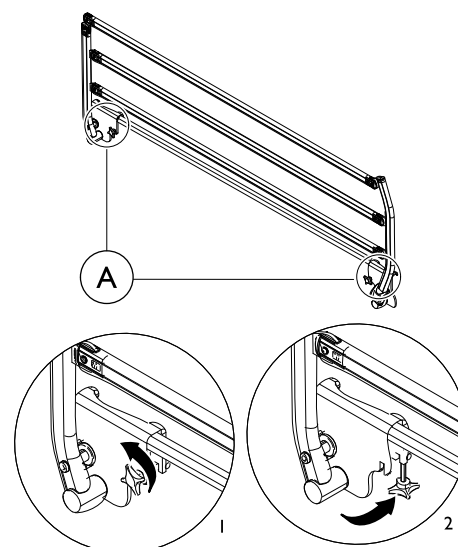
! **CAUTION!**
Risk of Injury
If the side rail is not locked properly it can fall down.
– Pull/push the top bar of the side rail to ensure that the locking system is properly engaged.

! **CAUTION!**
Risk of Injury
There is a risk of entrapment or squeezing while operating the side rail.
– Pay attention to your fingers and body parts of the patient.
– Never force or drop the side rail while handling it.

5.2.3 Verso HC side rail


Mounting the side rail

1. Unfold the side rail.
- 2.


Open the brackets by turning the finger screws **A** anti clockwise (1). Press the fingers screws down and fold them backwards completely out of the notches (2).

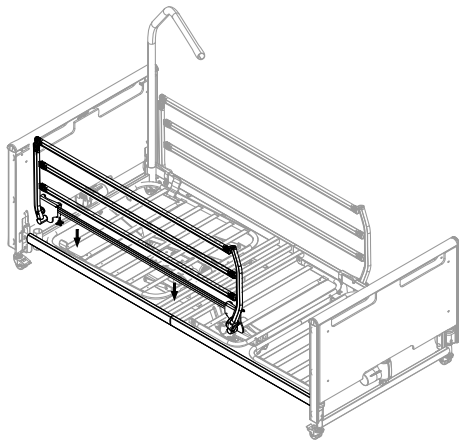3.

Press the U-formed side rail brackets onto the top frame.

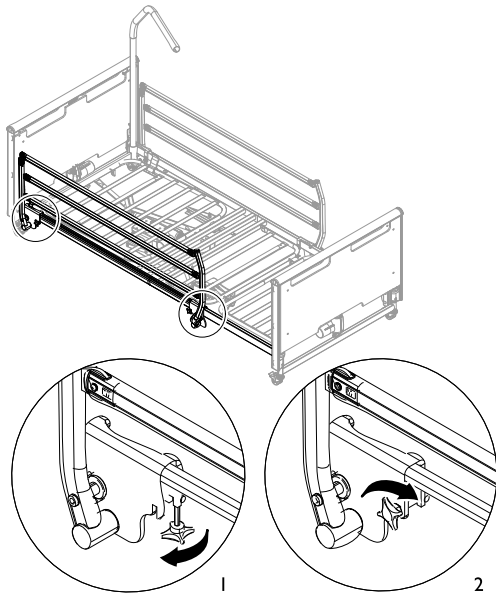
4.

Fold the finger screws forwards (under the top frame) and up into the notches (1). Close the brackets by turning the finger screws clockwise (2).

5. Make sure the finger screws are tightened and the side rail properly attached (not loose).

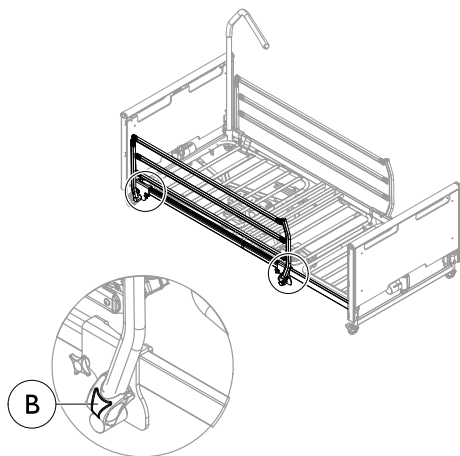Operating the side rail


Folding down the side rail

1. Hold the top bar of the side rail with one hand and push the release button (A) with the other hand.
2. Fold down the side rail sideways to its lowest position.

Folding up the side rail

1. Hold the top bar of the side rail and pull up until it locks with a clicking sound.

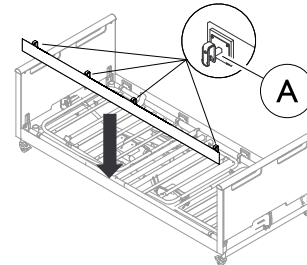5.3 Wooden side panel

Press the brackets (A) down, onto the top frame.

5.4 Support handles / Swivel handles

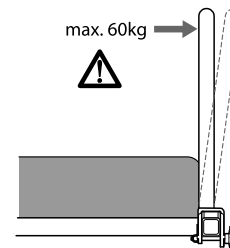
One or more support handles can be mounted on the top frame to support the user when getting in or out of bed.

WARNING!


Risk of serious injury

The support handles are not to be used as side rails; to keep the user from falling out of the bed – Only use the support handles as a support when getting in and out of bed.


Mounting the Support handles / Swivel handles

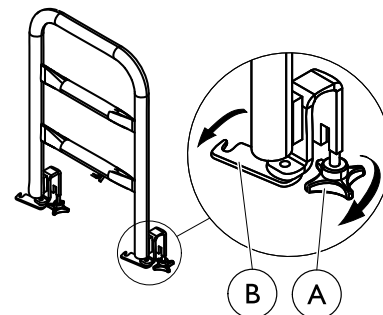1.


- a) Loosen the finger screws (A).
- b) Rotate the metal plates (B) out to the side.

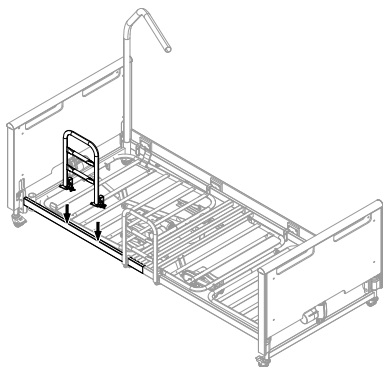2.


Press the brackets onto the top frame.

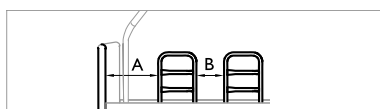WARNING!
Risk of squeezing, getting trapped or suffocated


Pay attention to the distance between support/swivel handle and bed end.
 Pay attention to the distance between two support handles.

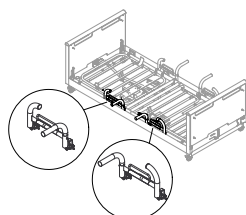- The safety distance **A** must be bigger than 390 mm.
- The safety distance **B** must be either 0–60 mm or 318 mm or bigger.
- Never use the support handles in combination with side rails.

The Swivel handle has to be mounted with the movable part towards the center of the bed.

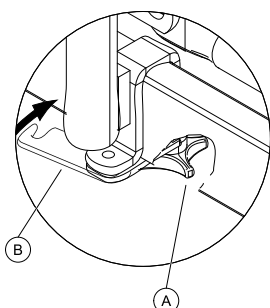3.


- a) Rotate the metal plates **B** back to close the brackets around the frame.
- b) Re-tighten the finger screws **A**.

Operating the Swivel handles


Open and close the handle

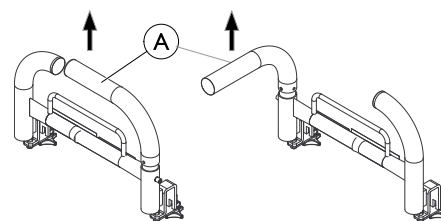1.


Pull the swivelling handle **A** up to disengage it.

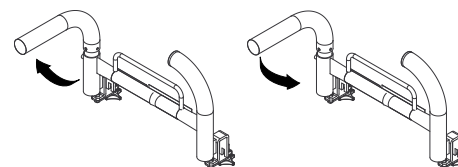2.

Rotate the handle inwards or outwards to/from the bed.

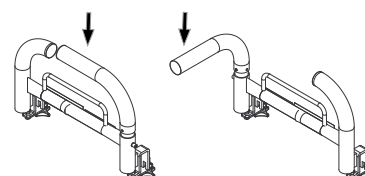
3.


Push the handle down to secure it. Make sure it is properly locked before using it as support.

5.5 Transport brackets

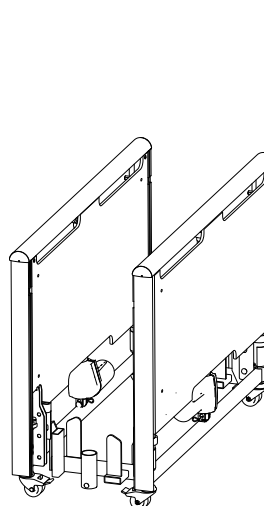
1.

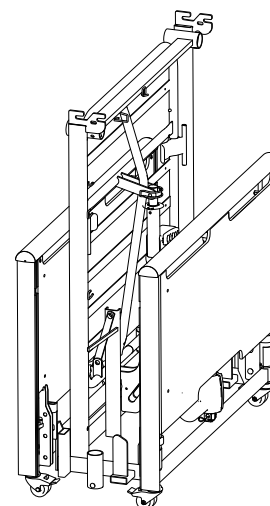
2.

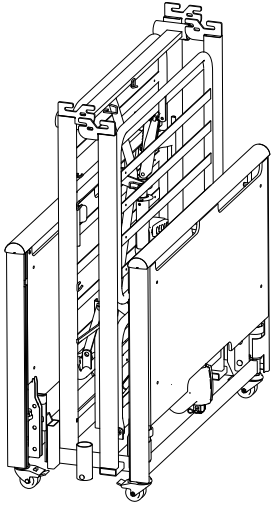3.

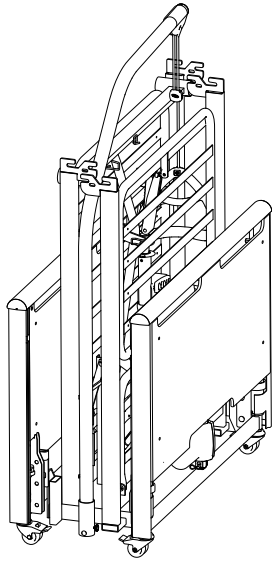4.

1. Attach both bed ends on the brackets.
2. Attach the mattress support head section.
3. Attach the mattress support leg section.
4. Place the lifting pole on the bracket.

6 Maintenance

6.1 General maintenance information

CAUTION!

Risk of injury or damage

- Do not perform any maintenance or service procedures while the product is in use.

Follow the maintenance procedures described in this manual to keep your product in continuous operation.

6.2 Daily inspections

WARNING!

Risk of injury or damage

- Damaged or worn parts can affect the safety of the product.
- The product should be checked each time it is used.
 - Do not use the product if damage is found or you question the safety of any part of the product. Contact your Invacare provider immediately and ensure the product is not used until repairs are made.

Daily Inspection Checklist

- Visually inspect the bed. Check all parts for external damage or wear.
- Check all hardware, attachment points, stressed parts and any pivot points for signs of wear, cracking, fraying, deformation or deterioration.
- Check the hand control and all actuators for smooth operation.

6.3 General servicing information

IMPORTANT!

- The mattress support must be supported during service inspections, to prevent accidental lowering.
- Only personnel who have received the necessary instructions or training may perform service and maintenance of the bed.
- After reconditioning the bed, or if bed functions change, service must be carried out according to the check list.

A service contract can be made in the countries, where Invacare has its own sales company. In certain countries Invacare offers courses in service and maintenance of the bed. Spare parts lists and additional user manuals are available from Invacare.

Before Use

- Ensure that all manual and electrical parts functions correctly and are in a secure state.

After three months

- Ensure that all manual and electrical parts are functioning, and tighten bolts, screws, nuts, etc.

Every year

- We recommend a safety test comprising the actuators' performance and mechanical state.

Every second year

- We strongly recommend a service according to the below checklist after two years of normal use and then every second year.

Actuators, hand control and control unit are serviced by exchanging the faulty part.

6.3.1 Checklist maintenance

Checkpoints

- Visual inspection of all parts of the bed (no deformations)
- Circlips, cotter pins and plastic fixing ring - properly locked and intact.
- Screws - tightened.
- Weldings - intact (no wear and tear)
- Castors (rolling smoothly)
- Castor fittings - tightened.
- Castor brakes - locking properly.
- All motors - running properly (with regular speed and at low noise).
- Wires - correctly wired and undamaged.
- Electric plugs - undamaged.
- Side rail locking and moving system - properly locking and running smoothly.
- Accessories - correctly assembled and correct function.
- Damaged coating - repaired.
- Leakage of grease - check for grease.

CAUTION!

Risk of injury or damage to property

- Grease leaks can lead to accidental falls or fire.
- Contact service provider if leaks of grease or other residue is detected.

6.4 Inspection after relocation - Prepare for new user

IMPORTANT!

- When the bed has been relocated; before given to a new user it has to be thoroughly inspected.
- Inspection must be done by a trained professional.
 - For regular maintenance see maintenance chart.

6.4.1 Checklist - After relocation

Checkpoints

- Check that the inserts connecting the two mattress support halves are fully inserted and locked.
- Check that the mattress support is correctly mounted to the bed ends and that the locking ring is engaged.
- Check the locking of the motors (pipe pins correctly mounted).
- Check the electronic wiring for the motors (wires not squeezed).
- Check that the enclosing of the motors are intact (no cracks that enables fluids to pierce).
- Check that the locking cam is properly mounted over the contacts in the control box.
- Check the sections of the mattress support by using the hand control to activate all functions of the moving parts.

- Check the function of the brakes.
- Check the locking function(s) of the side rails.
- Check that all bed slats are intact and not loose.

6.5 Cleaning and Disinfection

6.5.1 General safety information

CAUTION!

Risk of contamination

- Take precautions for yourself and use appropriate protective equipment.

CAUTION!

Risk of electric shock and product damage

- Switch off the device and disconnect from mains, if applicable.
- When cleaning electronic components consider their protection class regarding water ingress.
- Make sure that no water splashes to the plug or the wall outlet.
- Do not touch the power socket with wet hands.

IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

6.5.2 Cleaning Intervals

IMPORTANT!

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.
- Clean and disinfect the product
- regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

6.5.3 Cleaning Instructions

IMPORTANT!

- The product does not tolerate cleaning in automatic washing systems, with high-pressure cleaning equipment or steam.

Electrical components

- Method: Wipe off with a wet cloth or soft brush.
- Max. temperature: 40 °C
- Solvent/chemicals: Ordinary household cleaning agents and water.
- Drying: Wipe dry with a soft cloth

Metal components

- Method: Wipe off with a wet cloth or soft brush.
- Max. temperature: 40 °C
- Solvent/chemicals: Ordinary household cleaning agents and water.
- Drying: Wipe dry with a soft cloth

Metal components may also be cleaned with pressurized water, but not high pressure or steam.

Wood (including textile straps on side rails, if existing)

- Method: Wipe off with a wet cloth or soft brush.
- Max. temperature: 40 °C
- Solvent/chemicals: Ordinary household cleaning agents and water.
- Drying: Wipe dry with a soft cloth

Textiles (including upholstery and mattresses)

- See attached label on each product.

6.5.4 Disinfection Instructions

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

In Domestic Care

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

In Clinical and Long Term Care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

6.6 Lubrication

We recommend lubricating the bed according to the following instruction after service and cleaning or whenever it is necessary to reduce friction.

Part of bed	Lubrication method
Points of rotation in mattress support	Oil (medically clean)
Motor attachment points on mattress support	Oil (medically clean)
Motor bearings and tension rods	Oil (medically clean)

- The wooden side rails gliding system must not be lubricated with oil - otherwise the wooden bars will move sluggishly.
- For ordering of correct oil, contact your Invacare provider.

7 After Use

7.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *6 Maintenance, page 19*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

8 Troubleshooting

8.1 Troubleshooting electrical system

Symptom	Possible cause	Remedy
Mains indicator does not light up	Mains are not connected	Connect mains
	Fuse in the control unit is blown	* Replace the control unit
	Control unit is defective	* Replace the control unit
Mains indicator lights up, but the motor is not running. The relay in the control unit makes a clicking noise.	Motor plug is not fully inserted into the control unit.	Insert the motor plug properly into the control unit
	The motor is defective.	* Replace the motor
	Motor cable is damaged.	* Replace the cable
	Control unit is defective	* Replace the control unit
Mains indicator lights up, but the motor is not running. No relay sound is heard from the control unit.	Control unit is defective	* Replace the control unit
	Hand control is defective	* Replace the hand control
Control unit is in order except for one direction on one channel.	Control unit is defective	* Replace the control unit
	Hand control is defective	* Replace the hand control
Motor is running, but the piston rod does not move.	Motor is damaged	* Replace the motor
The motor cannot lift full load.		
Motor noise, but no movement of piston rod.		
Piston rod operates inwards and not outwards.		

* Service and maintenance of the bed must only be performed by personnel who have received the necessary instruction or training.

Risk of personal injury and damage to the product.

– The bed must be unplugged from the main power source before opening or repairing electrical parts.

WARNING!

Risk of personal injury and damage to the product.

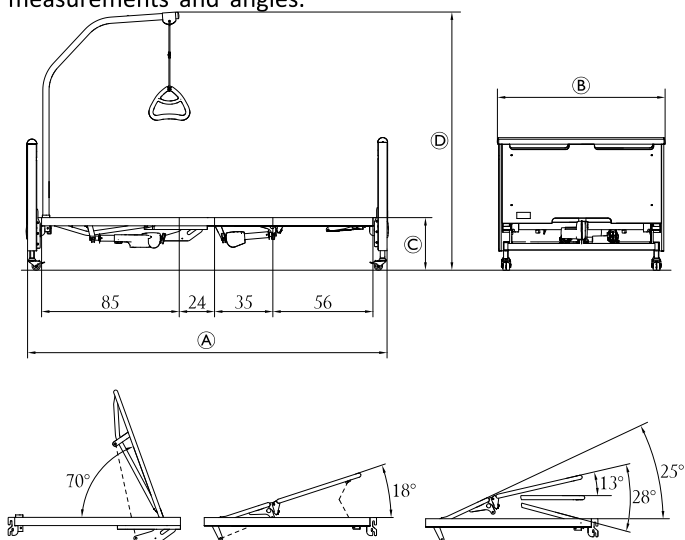
– The bed must be unplugged from the main power source before opening or repairing electrical parts.

9 Technical Data

9.1 Dimensions

All measurements are stated in cm. All angles are stated in degrees. All measurements and angles are stated without tolerances.


Invacare reserves the right to change the stated measurements and angles.

Etude Plus HC		
	Low	High
Ⓐ	219	219
Ⓑ	102	102
Ⓒ	20–60	27–67
Ⓓ	138–178	145–185

The angle between the lower leg section and horizontal is adjustable from 0° to 15°.

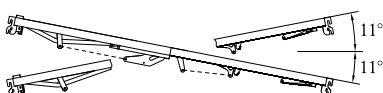Bed with tilt function

9.2 Weights

	Max. patient weight (provided that the weight of the mattress and the accessories do not exceed 35 kg)	165 kg
	Max. safe working load (patient + accessories)	200 kg
Etude Plus standard bed, complete, excluding accessories		77.3 kg
Etude Plus auto-regression, complete, excluding accessories		82.7 kg
Etude Plus bed end – 1 piece		17.9 kg
Mattress support, standard head end		24.1 kg
Mattress support with auto-regression, head end		27.9 kg

Mattress support with auto-regression, head end, without actuator and control box	24.1 kg
Mattress support, foot end	17.4 kg
Verso HC side rail – 1 piece	7.4 kg
Lifting pole	4.2 kg
Mattress support extension (10 cm)	3.2 kg
Mattress support extension (20 cm)	5.5 kg
Support handle 40 x 30 cm	1.6 kg
Support handle 40 x 40 cm	2.0 kg
Swivel handle	1.8 kg

9.3 Mattress dimensions

Side rail:	Mattress measures (in cm):			
	Min. height	Max. height	Min. width	Min. length
Verso HC	14	16	90	190

Min. density: 38kg/m³

9.4 Environmental conditions

	Storage and Transportation	Operation
Temperature	-10°C to +50°C	+5°C to +40°C
Relative humidity	20% to 90% at 30°C - not condensing	
Atmospheric pressure	800 hPa to 1060 hPa	

- Allow the product to reach operation temperature before usage:
- Warming up from minimum storage temperature might take up to 24 hours.
 - Cooling down from maximum storage temperature might take up to 24 hours.

9.5 Electrical system

Voltage supply	Uin = 120 Volt, AC, 60 Hz (AC = Alternating current)
Maximum current input	Iin max. = 1.7 A
Intermittent (periodic motor operation)	Int = 10%, max. 2 min / 18 min
Degree of protection	IPX4 – The electrical system is protected against water splashed against the components from any direction.
Insulation class	Class II equipment
Type B Applied Part	Applied Part complying with the specified requirements for protection against electrical shock according to IEC60601-1.
Sound level	45 to 50 dB (A)

- The bed has no isolator (main switch). If the bed needs to be electrically disconnected, it has to be unplugged from the mains socket.

10 Electromagnetic compatibility (EMC)

10.1 General EMC information

Medical Electrical Equipment needs to be installed and used according to the EMC information in this manual.

This product has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2 for Class B equipment.

Portable and mobile RF communications equipment can affect the operation of this product.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standard. To determine if the emission from this product is causing the interference, run and stop running this product. If the interference with the other device operation stops, then this product is causing the interference. In such rare cases, interference may be reduced or corrected by the following:

- Reposition, relocate, or increase the separation between the devices.

10.2 Electromagnetic emission

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group I	This product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This product is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Complies	

10.3 Electromagnetic Immunity

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines; 100 kHz repetition frequency ± 1 kV for input / output lines; 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line to line ± 2 kV line to earth	Mains power quality should be that of a typical commercial or hospital environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 0% U_T for 0,5 cycle at 45° steps 0% U_T for 1 cycles 70% U_T for 25 / 30 cycles < 5% U_T for 250 / 300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of this product requires continued operation during power mains interruptions, it is recommended that the product is powered from an un-interruptible power supply or a battery. U_T is the a. c. mains voltage prior to application of the test level.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 V 150 kHz to 80 Mhz 6 V in ISM & amateur radio bands	Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which this product is used exceeds the applicable RF compliance level above, this product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating this product. Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	10 V/m 80 Mhz to 2.7 GHz 385 MHz - 5785 MHz test specifications for immunity to RF wireless communication equipment refer to table 9 of IEC 60601-1-2:2014	Portable and mobile RF communications equipment should be used no closer than 30 cm to any part of this product including cables.

 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

10.3.1 Test specifications for immunity to RF wireless communications equipment

IEC 60601-1-2:2014 — Table 9

Test Frequency (MHz)	Band ^{a)} (MHz)	Service ^{a)}	Modulation ^{b)}	Maximum power (W)	Distance (m)	Immunity test level (V/m)
385	380 - 390	TETRA 400	Pulse modulation ^{b)} 18 Hz	1.8	0.3	27
450	430 - 470	GMRS 460, FRS 460	FM ^{c)} ± 5 kHz deviation 1 kHz sine	2	0.3	28
710 745 788	704 - 787	LTE Band 13, 17	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9
810 870 930	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation ^{b)} 18 Hz	2	0.3	28

Test Frequency (MHz)	Band ^{a)} (MHz)	Service ^{a)}	Modulation ^{b)}	Maximum power (W)	Distance (m)	Immunity test level (V/m)
1720 1845 1970	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation ^{b)} 217 Hz	2	0.3	28
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^{b)} 217 Hz	2	0.3	28
5240 5500 5785	5100 - 5800	WLAN 802.11 a/n	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9

 If necessary to achieve the immunity test level, the distance between the transmitting antenna and the Medical Electrical Equipment or System may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

^{a)} For some services, only the uplink frequencies are included.

^{b)} The carrier shall be modulated using a 50 % duty cycle square wave signal.

^{c)} As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

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