CBJH Control Unit

en	Replacing the batteries	Service Manual	1
cs	Výměna baterií	Servisní příručka	2
da	Udskiftning af batterier	Service manual	2
de	Auswechseln der Batterien	Service Handbuch	3
es	Sustitución de la baterías	Manual de servicio	3
et	Akude asendamine	Teenusejuhend	3
fi	Akkujen vaihtaminen	Huoltokäsikirja	4
fr	Remplacement des batteries	Manuel de maintenance	4
hu	Az akkumulátorok cseréje	Szervizkönyv	4

it	Sostituzione delle batterie	Manuale per la manutenzione	5
lt	Akumuliatorių keitimas	Techninis vadovas	5
lv	Akumulatoru nomaiņa	Apkopes rokasgrāmata	5
nl	De accu's vervangen	Servicehandleiding	6
no	Bytte av batteri	Servicehåndbok	6
pl	Wymiana akumulatorów	Instrukcja serwisowa	6
pt	Substituir as baterias	Manual de assistência	7
ro	Înlocuirea acumulatorilor	Manual de service	7
sl	Zamenjava akumulatorjev	Navodila za servisiranje	8
sv	Byta batterierna	Service manual	8

I General

en

I.1 Figures

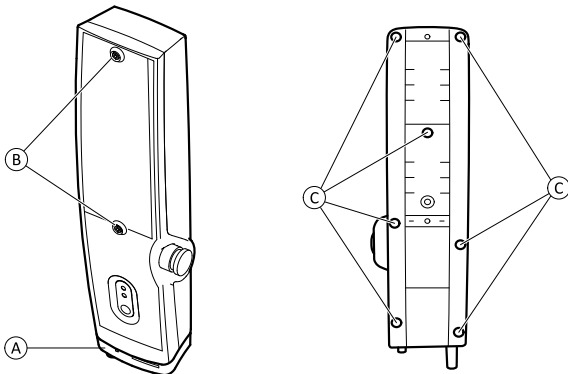
Figure 1

Figure 2

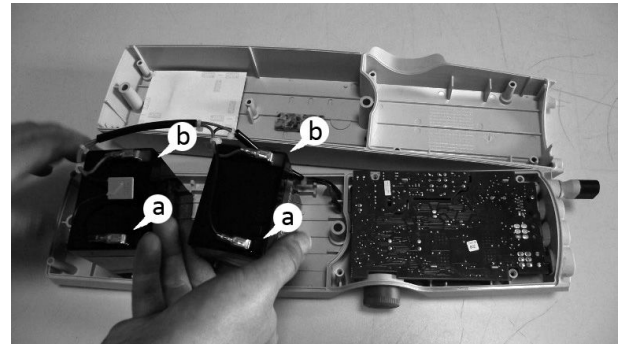
Figure 3

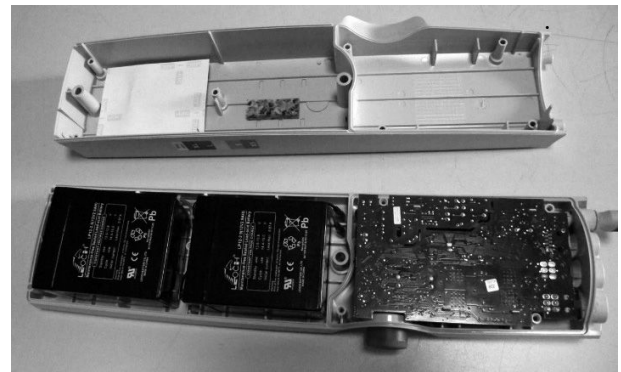
Figure 4

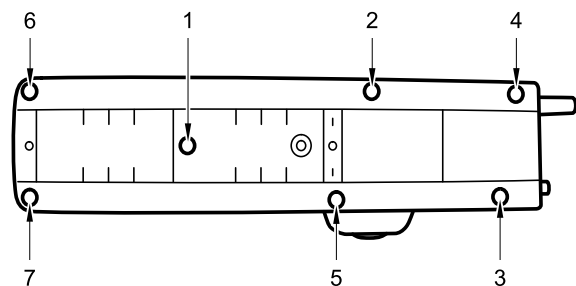
Figure 5

DEALER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.

Yes, you can.®

1.2 Introduction

This manual belongs to an Invacare accessory and it contains important information about handling and assembly. To ensure safety when using the product, read the user manual of the related product carefully and follow the safety instructions.

Find the user manual on Invacare's website or contact your Invacare representative (see addresses at the end of this manual).

In this manual, hazards are indicated by symbols. The symbols are accompanied by a signal word that indicates the severity of the risk.

2 Replacing battery

CAUTION!

Risk of injury

- Activate the emergency stop before removing the control unit.

IMPORTANT!

A short circuit can damage electrical components.

- Avoid electrostatic discharge (ESD) when working on electrical parts.

Torx screwdriver

1. Remove plastic cover (A) and remove all cables from control box.
2. Unscrew two screws (B) and remove the control unit from the lifter.
3. Unscrew seven screws (C) on the rear and remove back cover. See figure 1.
4. Remove both cables from each battery. See figure 2.
5. Replace the batteries.
6. Connect new batteries with the cables. Use the shorter cables for the closest battery. See figure 3.
 - a. black cables on black pin.
 - b. red cables on red pin.
7. Put all cables back into place and make sure no cables are squeezed. See figure 4.
8. Reinstall the back cover. Fix the seven screws in the order as described in figure 5 (Torque 1.5–1.6 Nm).
9. Reinstall the control unit to the lifter.

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All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare New Zealand Ltd