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**Yes, you can.®**

## Assembly instruction

Assembly of the hub and brake for self-propelling wheels Aquatec® Ocean

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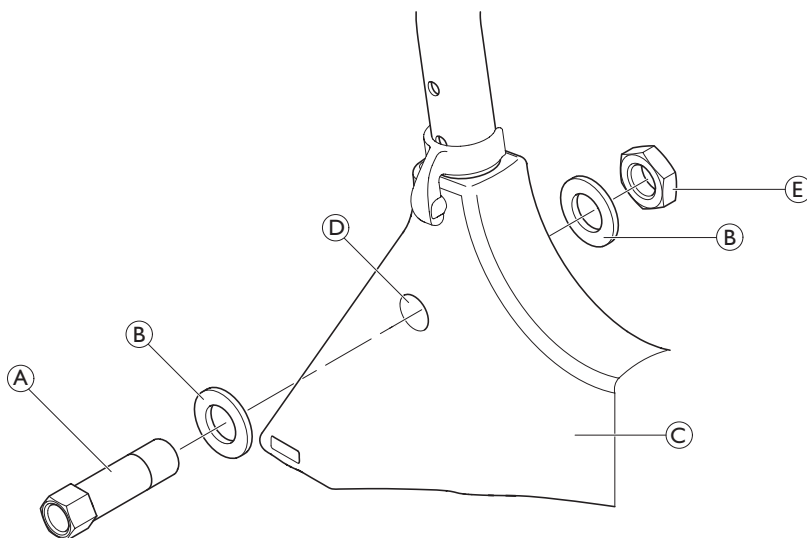
### Assemble the hub

EN

The procedure is the same for the right and left side.

1. Remove the dummy plugs from the holes Ⓓ of the side frame Ⓒ.
2. Put a washer Ⓑ on the hub Ⓐ.
3. Push the hub with the washer into the hole Ⓓ of the side frame.
4. From the inside, put a washer Ⓑ on the hub and screw on the nut Ⓔ.
5. Hold the bolt with a suitable tool and tighten the nut Ⓔ with a torque of 20 - 30 Nm.

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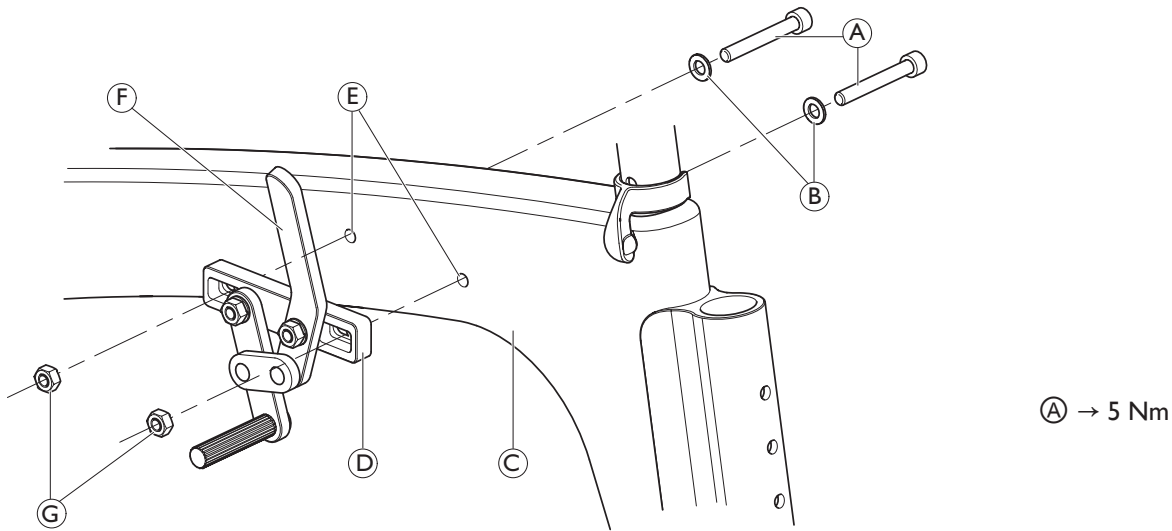
## Assemble brake

EN

The procedure is the same for the right and left side.

1. Remove the dummy plugs from the holes ⑤ of the side frame ③.
2. Put on the brake ④ with the lever ⑥ facing forward.
3. Put the washers ② on the bolts ① and put in the bolts from the inside.
4. Put in the nuts ③ and tighten the bolts ① with a torque of 5 Nm.
5. Place the 24" Wheel on the hub (→ Installing/Adjusting 24" Wheel and Quick-Release Axle, page 3).
6. Readjust brakes.

### IMPORTANT

▸ Check function of brakes regularly.

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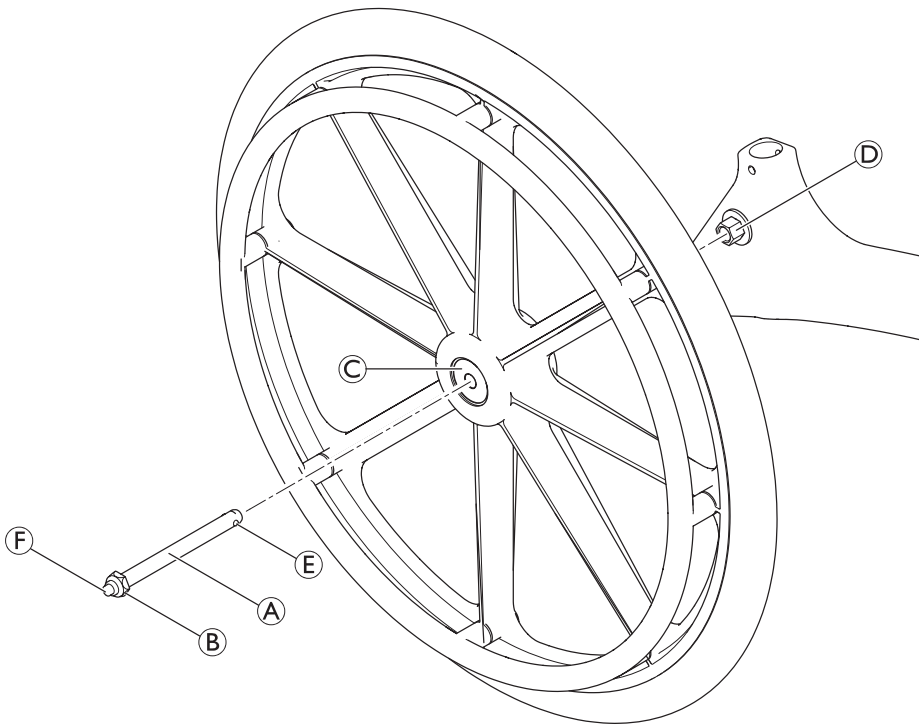
## Installing/Adjusting 24" Wheel and Quick-Release Axle

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### WARNING

- Make sure that the hub **D** is securely tightened before installing the 24" wheels.
- Make sure that both sprung bearings **C** are completely and securely in place.

1. Depress detent pin **F** in the quick-release axle **A** and slide axle through the 24" wheel and the hub.
2. Release detent pin ensuring that the locking pins **E** are fully released.

### IMPORTANT

- Make sure that the locking pins protrude past the end of the hub to fully release.
- Eliminate excessive play of the quick release axle.

3. Increase or decrease end play by adjusting the locknut **B** on the end of the quick-release axle.

### WARNING

- Make sure that the detent pin **F** is fully released before operating.

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Aquatec Operations GmbH