Invacare® Alizé

H3000

en	Bath Tub Swivel Seat User Manual	3
de	Schwenksitz für Badewanne Gebrauchsanweisung	5
fr	Siège pour baignoire pivotant Manuel d'utilisation	8
it	Sedile girevole per vasca Manuale d'uso	10
es	Silla giratoria para bañera Manual del usuario	13
pt	Assento de banheira rebatível Manual de utilização	16
nl	Draaibare badkuipzitting Gebruiksaanwijzing	18
sv	Vridbar badkarssits Bruksanvisning	21
fi	Käännettävä kylpyistuin Käyttöohje	24

<p>Fig. 1</p>	<p>Fig. 2</p>	<p>Fig. 3</p>
<p>Fig. 4</p>	<p>Fig. 5</p>	

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

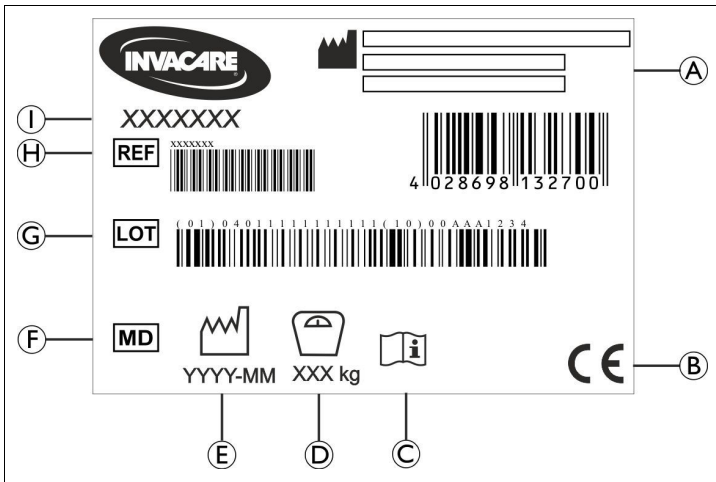
Fig. 6

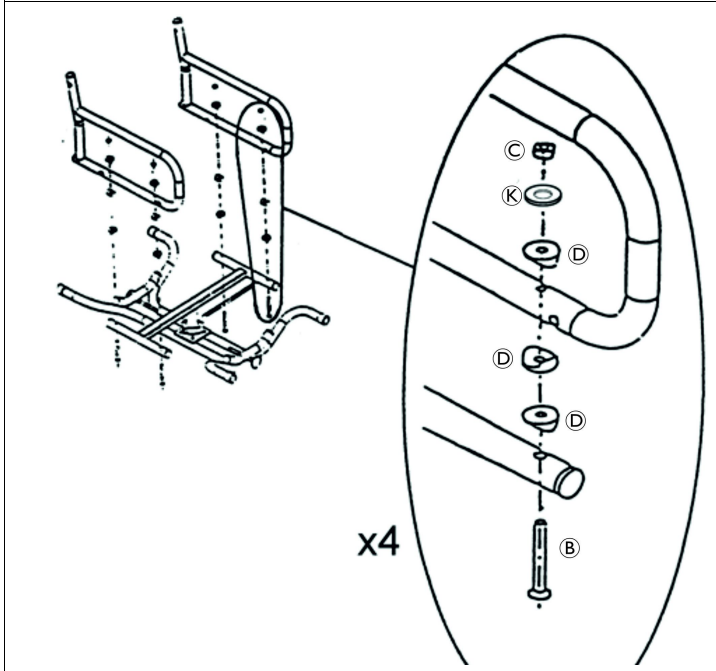
Fig. 7

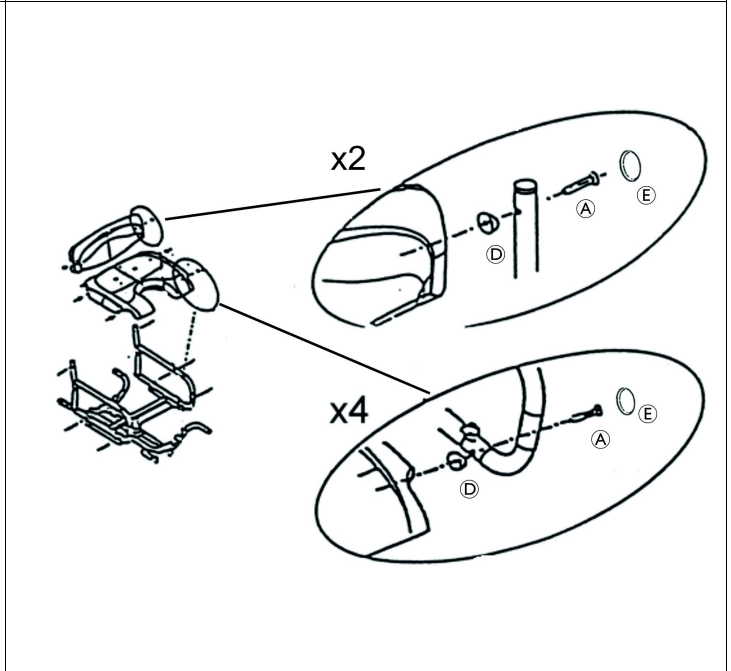
Fig. 8

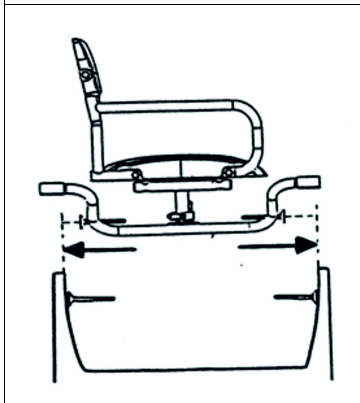
Fig. 9

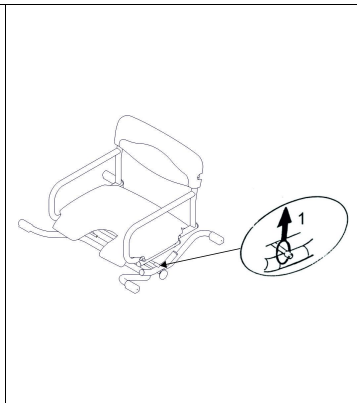
Fig. 10

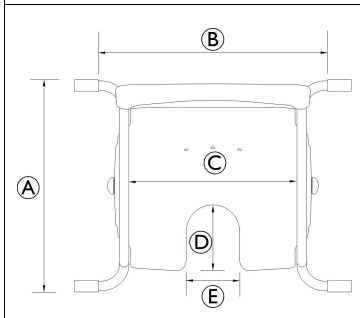
Fig. 11

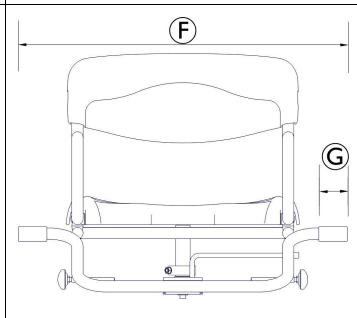
Fig. 12

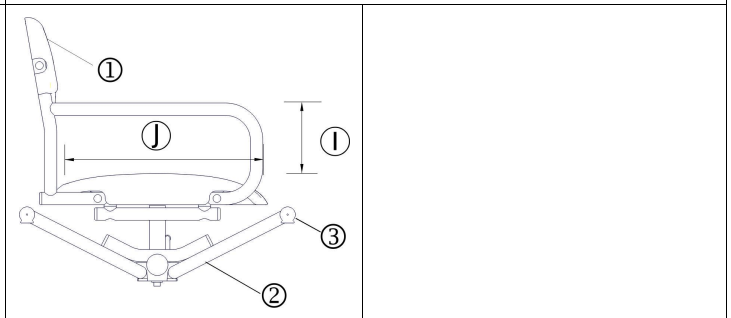
Fig. 13

1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

3.1 Product-specific Standards

The product has been tested and conforms to EN 12182 (Assistive products for persons with disability) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

6 Safety

6.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of Falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Do not stand on this product.
- Do not use as a climbing aid.
- Do not use or store the product outdoors.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

6.2 Labels and Symbols on the Product (Fig. 8)


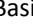
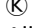
(A)	Manufacturer	(F)	Medical Device
(B)	European Conformity	(G)	Lot number
(C)	Read user manual	(H)	Reference number
(D)	Load capacity	(I)	Product designation
(E)	Production date		

The identification label is attached to the frame of the product.

7 Scope of Delivery

- (A) Screws (M6 x 50 mm) (6x)
- (B) Screws (M6 x 70 mm) (4x)
- (C) Nut (M6) (4x)
- (D) Saddle spacer (18x)
- (E) Covering cap for screw (6x)
- (F) Backrest
- (G) Seat plate

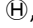-   Frame with arm rest (left and right)
-  Basis
-  Washers (4x)
- Allen key no. 5
- User manual

8 Intended Use

This product is to be used as a shower aid.

Users are any adults and adolescents with restricted or limited mobility (e.g.: legs and/or hips). If used without an assistant, users must be able to keep their own balance.

The product is intended exclusively for home use and is not intended for intensive use in institutions.

Indication: Restricted or limited mobility (e.g.: legs and/or hips) or having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended.

The maximum user weight is 120 kg

9 Product Description

Invacare® Alizé H3000 is a bathtub swivel seat. The seat can be rotated and it is supported by four feet being placed on the bathtub rim.






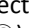
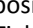
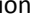
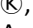
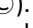
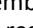
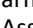
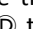
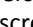
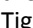
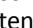
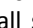
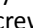
10 Assembling the Product (Fig. 7 — 10)

CAUTION!

Risk of Trapping

- Take care during assembly.

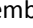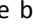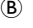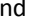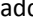1. Assemble both frames with armrest   with screws  and saddle spacers  to the basis  ensuring the correct positioning of parts ( Tube,  Tube,  Tube,  Tube,  ). Tighten the screws by hand. (Fig. 7)
2. Assemble the seat  with screws  to the frame with arm rest  . (Fig. 8) Tighten the screws by hand.
3. Assemble the backrest  with screws  and saddle spacers  to the frames with arm rest  and . Tighten the screws by hand. (Fig. 8)
4. Tighten all screws with the Allen key no. 5. Install the covering caps  on the screw heads.
5. Fit the basis into your bath tub by adjusting the setting screws. (Fig. 9) The four feet must lie safely on the rim of the bath tub. Make sure the swivelling range is free to avoid collisions.

CAUTION!

Risk of Injury

- Ensure the feet cannot slide into the bath tub on one side by pushing the installed product to one rim of the bath tub. The feet on the other side must still lie safely on the rim of the bath tub.

6. The swivelling mechanism is released by pulling up the lever. The seat can now be swivelled by 360°. (Fig. 10)

11 Maintenance

11.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

11.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
 - All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking it into use again.

Cleaning Intervals

IMPORTANT!

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination. Clean and disinfect the product
 - regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

Cleaning Instructions

IMPORTANT!

- Clean by hand
- Maximum cleaning temperature 60 °C

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

12 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *11 Maintenance, page 4*.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

13 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

14 Technical Data (Fig. 11 — 13)

Dimensions			
Ⓐ	556 mm	Ⓕ	725 mm
Ⓑ	600 mm	Ⓖ	65 mm
Ⓒ	440 mm	Ⓖ	163 mm
Ⓓ	170 mm	Ⓙ	400 mm
Ⓔ	140 mm		

Weights	
Alizé H3000 Bath tub swivel seat	9.8 kg
Maximum user weight	120 kg

Materials			
①	ABS	③	Steel, powder-coated
③	TPE		

All product components are corrosion resistant and do not contain natural rubber latex.

15 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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- © istuinlevy
Invacare Limited