

Invacare® NordBed™

NordBed™ Ultra

NordBed™ Optimo

NordBed™ Optimo Wide

en Bed

User Manual



This manual **MUST** be given to the user of the product.
BEFORE using this product, this manual **MUST** be read and
saved for future reference.



Yes, you can.

Contents

1 General	3		
1.1 Introduction	3		
1.1.1 Symbols in this Document	3		
1.2 Service Life	3		
1.3 Compliance	3		
1.3.1 Product-specific standards	3		
1.4 Warranty Information	3		
1.5 Limitation of Liability	3		
2 Safety	4		
2.1 General safety information	4		
2.2 Mattresses	5		
2.3 Under Bed Clearance	5		
2.4 Safety information on electromagnetic interference	5		
2.5 Labels and symbols on the product	6		
2.5.1 Identification Label	6		
2.5.2 Other Labels	7		
3 Product Overview	8		
3.1 Intended Use	8		
3.1.1 Intended Operator	8		
3.2 Main parts of bed	8		
3.3 Options	8		
4 Usage	10		
4.1 General Safety Information	10		
4.1.1 Mattress retainers	10		
4.2 Setup	10		
4.3 Castors and brakes	10		
4.3.1 Central brake system	10		
4.4 Hand Control	11		
4.4.1 Locking function	11		
4.5 Hand Control with LCD (optional)	12		
4.5.1 Locking function	13		
4.5.2 Storing preferred positions	13		
4.6 Basic Hand Control	14		
4.6.1 Locking function	14		
4.7 Operating side rails	15		
4.7.1 Operating the Ane side rail	15		
4.7.2 Operating the Freya / Embla side rail	15		
4.7.3 Operating the Torill side rail	16		
4.8 Lifting pole	16		
4.8.1 Place the lifting pole	16		
4.8.2 Adjusting the handle height	17		
4.9 Emergency positions	17		
4.9.1 Emergency release of a mattress support section	17		
4.10 Transportation and Storage	17		
5 Maintenance	18		
5.1 General maintenance information	18		
5.1.1 Service interval	18		
5.2 Daily inspections	18		
5.3 Cleaning and Disinfection	18		
5.3.1 General Safety Information	18		
5.3.2 Cleaning Intervals	18		
5.3.3 Cleaning Instructions	18		
5.3.4 Disinfection Instructions	19		
6 After Use	20		
6.1 Disposal	20		
6.2 Reconditioning	20		
7 Troubleshooting	21		
7.1 Troubleshooting	21		
8 Technical Data	22		
8.1 Materials	22		
8.2 Dimensions bed	22		
8.3 Maximum loads	23		
8.4 Weights	23		
8.5 Dimensions side rails	24		
8.6 Allowed mattress sizes	25		
8.7 Electrical system	25		
8.8 Environmental conditions	26		
9 Electromagnetic compatibility (EMC)	27		
9.1 General EMC information	27		
9.2 Electromagnetic emission	27		
9.3 Electromagnetic Immunity	28		
9.3.1 EMC test specifications	29		

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Only use this product if you have read and understood this manual. Seek additional advice from a healthcare professional who is familiar with your medical condition and clarify any questions regarding the correct use and necessary adjustment with the healthcare professional.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

Previous product versions may not be described in this manual's current revision. If you require assistance, please contact Invacare.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.1.1 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. This document is printed in greyscale. For your information, the safety messages have the following colour coding according to ANSI Z535.6: Danger (Red), Warning (Orange), Caution (Yellow) and Notice (Blue). See the information below for definitions of the signal words.



WARNING!

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.



CAUTION!

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.



NOTICE!

Indicates a hazardous situation that could result in damage to property if it is not avoided.



Tips and Recommendations

Gives useful tips, recommendations, and information for efficient, trouble-free use.

1.2 Service Life

The expected service life of this product is ten years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

1.3.1 Product-specific standards

The product has been tested and conforms to IEC 60601-2-52 (Particular requirements for the basic safety and essential performance of medical beds) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

1.4 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 General safety information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section.



WARNING!

Risk of injury or damage

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, Invacare provider or qualified technician before attempting to use this product.
- Do not make any unauthorized alterations or modifications to the product.



WARNING!

Risk of entrapment / suffocation

There's a risk of entrapment / suffocation between mattress support, side rail and bed end or between moving parts and objects placed nearby the bed.

- The bed must not be used by persons under 12 years of age, or by persons with a body size equivalent to an average 12 years old or smaller.
- The bed, in combination with side rails must not be used by persons having a physical size less than 146 cm, a weight less than 40 kg or a body mass index (BMI) less than 17.
- Due to mattress compression, an increased risk may occur over time. Periodically monitor gaps between the bed, mattress and/or side rail. Replace mattress if the gaps may lead to entrapment.



WARNING!

Risk of slipping through the openings

The bed fulfils all requirements regarding maximum distances. However, it is possible that persons with small body dimensions slip through the openings between the side rails or through the opening between the side rail and the mattress support.

- Pay special attention, if the bed is used for the care of persons with small body dimensions.



WARNING!

Risk of injury or damage

Inappropriate handling of cables, incorrect connections and use of unauthorized equipment can cause electrical shock and product failure.

- Do not kink, shear or otherwise damage the cables of the product.
- Make sure that no cables are jammed or damaged, when the product is used.
- Disconnect the plug from the mains before moving the bed.
- Make sure the wiring is correct and connections are proper.
- Do not use unauthorized equipment.



WARNING!

Risk of tripping, entanglement or strangulation

Improper routing of cables may cause tripping, entanglement or strangulation hazard.

- Ensure all cables are routed and secured properly.
- Ensure there are no loops of excess cable extending away from the product.



WARNING!

Risk of injury or damage

Ignition sources can cause burns or fire.

- Bed must be placed with a safety space from possible ignition sources (heater, fireplace, etc.)
- The patient and assistants must not smoke while using or operating the bed.



WARNING!

Risk of injury or damage

To avoid injury or damage when operating the product:

- Close supervision is necessary when the product is used near children or pets.
- Do not let children play with the product.



CAUTION!

- For a person entering or exiting the bed, always lower the bed to an appropriate height. For entering or exiting the bed, the backrest section of the mattress support can be used to assist the positioning of a person's back in a more seated position. The backrest section of the mattress support is not intended to support and/or raise the full body of a person. Make sure the patient is laid down with his back at the centre of the backrest section. While the backrest section is being raised, the allowed maximum weight for the backrest section is 45% of the maximum safe working load.
- Always lower the bed to the lowest position before leaving a person in the bed unattended.
- Make sure that there is nothing under, over, or near the bed that can obstruct the height adjustment, like for example furniture, lifts or window frames.

**CAUTION!**

Non-original or wrong accessories may affect the function and safety of the product.

- Only use original accessories for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available accessories or contact your local Invacare representative. See addresses at the end of this document.

**CAUTION!****Risk of injury or damage**

The product can get hot when exposed to sunlight or other heat sources.

- Do not expose the product to direct sunlight for prolonged periods.
- Keep the product away from heat sources.

**CAUTION!**

There is a risk of entrapment of fingers in the bed moving parts.

- Pay attention to your fingers.

**NOTICE!**

The bed has no isolator (main switch). If the bed needs to be electrically disconnected, it has to be unplugged from the mains socket.

- Always place the bed in a way so the mains socket is easily accessible to unplug the bed.

**NOTICE!**

Accumulation of lint, dust and other dirt can impair the product.

- Keep the product clean.

2.2 Mattresses


**WARNING!****Safety aspects regarding combination of side rails and mattresses:**

To get the highest possible safety level, when using side rails on the bed, the minimum and maximum measures for mattresses, must be respected.

- For correct mattress measures see mattress table in chapter 8 *Technical Data*, page 22.

**WARNING!****Risk of entrapment and/or suffocation**

- The user could get trapped and/or suffocate, if the horizontal space, between the mattress side and the inside of the side rail, is too big. Follow the minimum width (and length) of mattresses in combination with a side rail, as stated in the mattress table in chapter 8 *Technical Data*, page 22.
- Be aware that using very thick or soft mattresses (low density), or a combination of these, increases the risk.

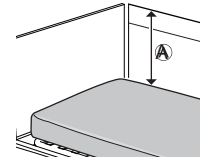**WARNING!****Risk of falling**

The user can fall over the edge and get seriously injured, if the vertical distance (A) between the top of the mattress and the edge of the side rail/bed end, is too short. See image above.

- Always keep a minimum distance (A) of 22 cm.
- Follow the maximum mattress height in combination with the side rail as stated in the mattress table in chapter 8 *Technical Data*, page 22.

2.3 Under Bed Clearance

**NOTICE!**


Nordbed Optimo and Nordbed Optimo Wide with 100 mm castors can have limited space under the bed. With the bed lowered to heights of 400 mm or less, the base of some mobile patient lifts might not fit under the bed. It might be required to use mobile patient lifts with a smaller base or other transfer methods.

2.4 Safety information on electromagnetic interference

**WARNING!****Risk of malfunction due to electromagnetic interference**

Electromagnetic interference between this product and other electrical equipment can occur and disturb the electrical adjustment functions of this product. To prevent, reduce or eliminate such electromagnetic interference:

- Only use original cables, accessories and spare parts, to not increase electromagnetic emission or reduce electromagnetic immunity of this product.
- Do not use portable radio frequency (RF) communications equipment closer than 30 cm to any part of this product (including cables).
- Do not use this product near active high-frequency surgical equipment and the RF shielded room of a system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.
- If disturbances occur, increase the distance between this product and the other equipment or switch it off.
- Refer to the detailed information and follow the guidance in chapter 9 *Electromagnetic compatibility (EMC)*, page 27.






WARNING!
Risk of malfunction due to electromagnetic interference

Do not use this bed adjacent to or stacked with other electrical equipment, than specified in the following, as it could result in improper operation. If such use is necessary, the bed and the other equipment must be closely observed to verify that they are operating normally.

This bed can be used together with Invacare approved accessories and medical electrical equipment connected to the heart (intracardially) or blood vessels (intravascularly) provided that following points are respected:

- Medical electrical equipment should not be fixed on the bed's metallic accessories such as side rails, lifting pole, drip rod, bed ends, etc.
- The medical electrical equipment power supply cord should be kept clear of the accessories or any other moving parts of the bed.

	Type B applied part
	Do not dispose of this product in normal household waste
	European Conformity

Abbreviations for technical data:


- Iin = Incoming Current
- Uin = Incoming Voltage
- Int. = Intermittence
- AC = Alternating Current
- Max = maximum
- min = minutes

For more information about technical data, refer to *8 Technical Data, page 22*.


2.5 Labels and symbols on the product

2.5.1 Identification Label


The identification label is placed on the frame of the bed and contains the main product information, including technical data.




Invacare Rea AB
Växjövägen 303
34375 Dlö - Sweden



YYYY-MM



MD




REF

XXXXXXXX


SN


XXXXXXXXXXXX


UDI




(01)XXXXXXXXXXXX(21)XXXXXXXXXXXX





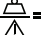








 = xxx kg

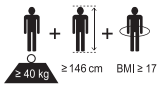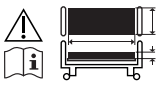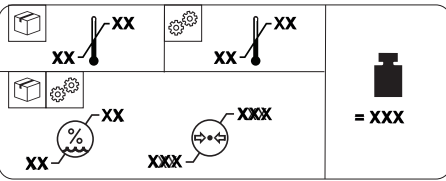 = xxx kg







2.5.2 Other Labels

	
<p>Read the user manual before using this product and follow all instructions for safety and use.</p>	
	
	<p>Definition of min. weight, min. height and min. body mass index of an adult user.</p> <p>See 3.1 Intended Use, page 8.</p>
	<p>Refer to user documentation for the correct mattress measures.</p> <p>See 8 Technical Data, page 22.</p>
	
	<p>Total weight of the product with the maximum safe working load applied</p>
	<p>Temperature limit</p>
	<p>Humidity limitation</p>
	<p>Atmospheric pressure limitation</p>
	<p>Transport and storage conditions</p>
	<p>Operation conditions</p>
	
<p>Indicates, that the bed can be used with automatic washing systems. (Only present on washable versions of this product)</p>	
	
<p>Marks out the location of connection for means of potential equalization. (Not present on all versions of this product)</p>	

Label on Ane side rails

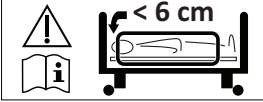
<p>Indicates the maximum space between side rail and the head bed end.</p> <p>See the mounting instruction in the service manual of the bed or the instruction delivered with the side rail.</p>

Label on Torill side rails

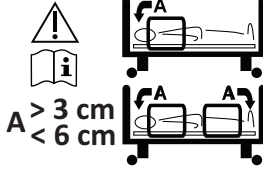
<p>Indicates the allowed mounting positions and spaces between the side rails and the bed ends, when only one side rail is mounted on the side of the bed or when two side rails are mounted on the same side of the bed.</p> <p>See the mounting instruction in the service manual of the bed or the instruction delivered with the side rail.</p>

3 Product Overview

3.1 Intended Use

The medical bed is electrically operated, adjustable in height and features a profiling support surface intended to be used in combination with a mattress. Medical beds are designed for support of the patient and used for the mitigation, relief, and recovery from certain medical conditions. The medical bed also supports and eases the caregivers' working environment.

The Medical bed is intended to be used on:

- *Application environment 3*; Long-term care in a medical area where medical supervision is required and monitoring is provided if necessary and medical electrical equipment used in medical procedures may be provided to help maintain or improve the condition of the patient.
- *Application environment 4*; Care provided in a domestic area where medical electrical equipment is used to alleviate or compensate for an injury, disability or disease.

Intended Users

A person with a medical condition that needs to be cared for in bed.

Indications

The bed is intended for adult users, having a physical size equal to or more than 146 cm, a weight equal to or more than 40 kg and a body mass index (BMI) equal to or more than 17.

The bed is intended for indoor use only.

Maximum user weight and safe working load are listed on the product label and in the technical data section in this manual.

Contra-indications

The bed is not intended for transportation of users. It is mobile within a room with a user in it. Castors are lockable.

The bed is not intended for persons with a psychosocial disability.



WARNING!

Any other or incorrect use could lead to hazardous situations.


Invacare accepts no liability for any use, change or assembly of the product, other than stated in this user manual.

3.1.1 Intended Operator

A healthcare professional or private person who has received proper training is the intended operator of this product.

The hand control can also be operated by the occupant of the bed.

3.2 Main parts of bed

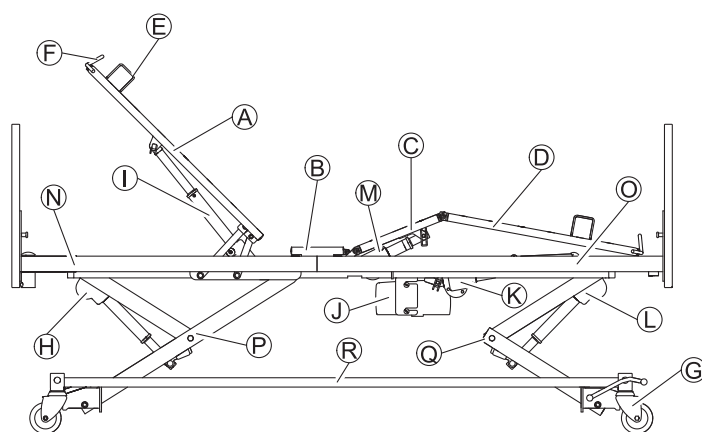

Parts of standard bed:

- Ⓐ Back section
- Ⓑ Seat section
- Ⓒ Thigh section
- Ⓓ Leg section
- Ⓔ Mattress retainer
- Ⓕ Mattress support extension
- Ⓖ Castor with central brake pedal
- Ⓗ Height adjustment actuator, upper half
- Ⓘ Back section actuator
- Ⓝ Control unit
- Ⓚ Leg section actuator
- Ⓛ Height adjustment actuator, lower half
- Ⓜ Thigh section actuator
- Ⓡ Mattress support platform, upper half
- Ⓢ Mattress support platform, lower half
- Ⓟ Lifting mechanism, upper half
- Ⓠ Lifting mechanism, lower half
- Ⓡ Base frame

3.3 Options



Due to regional differences, refer to your local Invacare website or catalog for details about available accessories or contact your Invacare provider.

Side rails

Side rails are intended to be operated by a person standing beside the bed and to reduce the risk for the user of unintentionally falling out of the bed, when locked in their uppermost position.

- Freya – Collapsible full length wood side rail (standard, 10 cm extended or 20 cm extended)
- Embla – Collapsible full length aluminum side rail (standard or extendable)
- Torill – Telescopic split side rail (75 cm or 96 cm length)
- Ane – Collapsible 3/4 length steel side rail (standard or high)

- Ane (83/85) – Collapsible 3/4 length steel side rail, for narrow mattresses on 90 cm wide beds only (standard or high)
- Height extenders for full length side rails – mounting positions for 6 or 10 cm height increase (for Freya or for Embla)
- Length extensions for Torill – for Torill on beds with extended mattress supports (10 cm or 20 cm length)
- Covers for side rails – textile covers for the different side rails (net or padded)

For more details on side rails see *8 Technical Data, page 22*.

Bed ends & side panels

Bed ends are intended to provide an esthetic and physical barrier at the head end and the foot end of the bed. Bed ends with guideways are required to mount full length side rails.

- Astrid, Nora and Una – bed ends with or without guideways for full length side rails
- Inga, Runa and Nanna – bed ends without guideways for full length side rails
- Wooden side panel – esthetic cover for the side of the bed
- Soft covers for bed ends – textile cover with padding for bed ends
- Soft covers for side panels – textile cover with padding for side panels

For more details on bed ends see *8 Technical Data, page 22*.

Repositioning aids

Repositioning aids are intended to provide support to the user when getting in or out of the bed (support handles and stand-up support) or when changing position inside the bed (support handles, lifting pole and sit-up support).

- Fixed support handles – fixed handles to be mounted on the side of the bed (short or long)
Not to be used with Ane and Torill side rails.
- Swiveling support handles – swiveling handles to be mounted on the side of the bed (left or right)
Not to be used with Freya/ Embla, Ane and Torill side rails or side panels.
- Sit-up support – adjustable handle to be mounted on the side of the bed (left or right)
Not to be used with Ane and Torill side rails.
- Stand-up support – handle to be mounted on one of the lifting pole tubes at the head end of the bed
- Lifting pole – to be mounted on one of the lifting pole tubes at the head end of the bed (fixed or revolving)

Other

- Distance bumpers – buffers mounted on the bed's base frame
- Fender Wheels – buffers mounted on bed ends
- Battery backup – mounted on the base frame for short term operation without mains power supply
- Holder for hand control – to be mounted on the side of the bed
- LCD hand control – hand control with LCD display
- Wireless hand control – Infrared hand control and receiver
- Rheumatic hand control – Hand control for people with rheumatism
- Y-cable – to use two hand controls on the same bed
- Soft Tilt – Patient repositioning system mounted on the mattress platform.

4 Usage

4.1 General Safety Information


WARNING!
Risk of personal injury and damage to property.

- The bed must be placed so that the height adjustment is not obstructed by, for example, lifts or furniture.
- Take care that no body parts are being squeezed between fixed parts (such as side rails, bed ends etc) and moving parts.
- The hand control must not be used by children.


WARNING!
Risk of injury

If the bed is used by confused or restless users or users with spasms:

- either, when the bed is equipped with a lockable hand control, lock the hand control functions,
- or make sure that the hand control is out of reach for the user.

4.1.1 Mattress retainers


WARNING!
Risk of injury

Without properly installed mattress retainers, the mattress can slip sideways and cause the user to fall out of the bed.

- Never use the bed without mattress retainers.
- Always make sure the mattress retainers are properly installed and facing upwards.
- Only use the original mattress retainers.

4.2 Setup


NOTICE!

- Unpacking and setup of the bed must be performed by a qualified technician as described in the service manual.
- Accessories, delivered with the bed, must be assembled by a qualified technician as described in the service manual or in the documentation, delivered with the accessory.

Scope of delivery

- Complete bed with castors, control box and mattress support

Options, like bed ends, side rails, lifting pole etc. are delivered apart.

Before taking the bed into use

1. Move the bed to its correct position.
2. Connect the bed to the mains.

4.3 Castors and brakes


CAUTION!
Risk of trapping/squeezing

All brakes are foot-operated.

- Do not release the brake with the fingers.


CAUTION!
Risk of injury

User can fall, while entering or exiting the bed, if brakes are not locked.


- Always lock the brakes before the user is moving in or out of bed or when attending to the user.




Castors might make marks on different types of absorbing floor surfaces - such as untreated or badly treated floors. To prevent marks, *Invacare®* recommends to place a suitable kind of protection between the castors and the floor.

4.3.1 Central brake system

All the wheels are locked by locking one of the pedals.

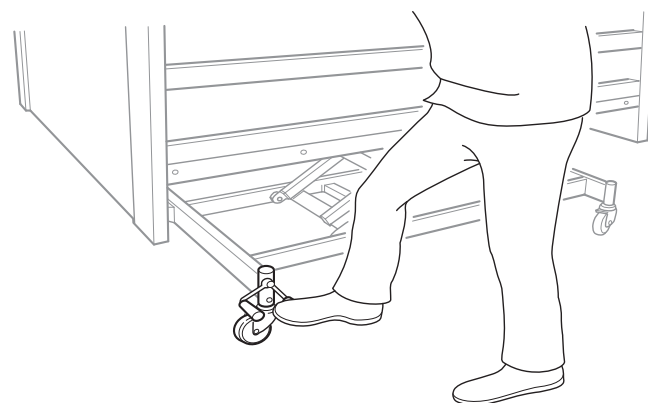


Always stand in the middle of the bed's long side when locking the brakes.

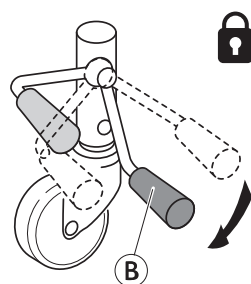

1. Lock the brakes:

Step on the red pedal **B**.

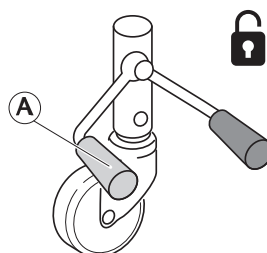


2. Unlock the brakes:

Step on the green pedal **A** until the brake is in **neutral position** = Both pedals on the same level.


Steerable castor (optional)

The bed with central braking system may be equipped with a steerable castor operated with the central braking pedal.

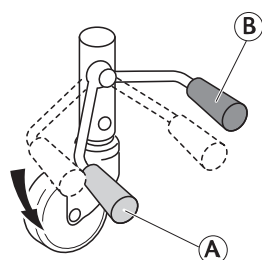

1. Activating the steering:
When the brake is in neutral position, step on the green pedal (A).
2. Deactivating the steering:
When steering is activated; step on the red pedal (B), until the brake is in neutral position.

4.4 Hand Control

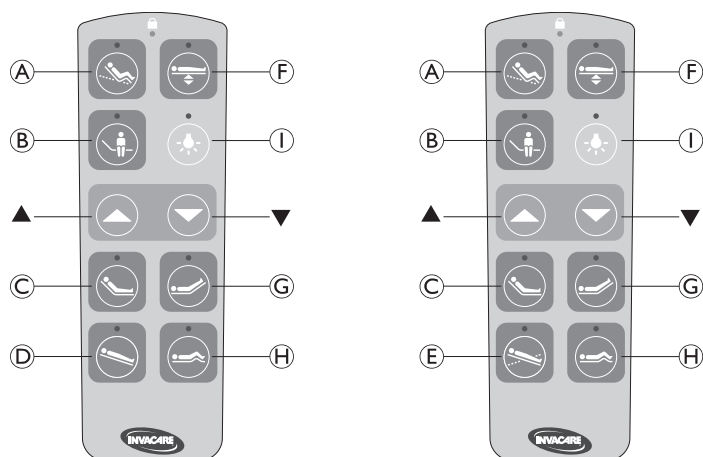

The hand control is equipped with buttons for electrical functions (A to H), up and down buttons (▲ and ▼) to control the selected function and optionally with a button for an under bed light (I).

Depending on the bed configuration, the hand control is equipped either with the Anti-Trendelenburg function (D) or with the Trendelenburg / Anti-Trendelenburg function (E).

All symbols light up when pressing any button. The selected function is indicated by a blue LED light above the button.

ErgoMove™



1. Press button (A) to select the function.
2. Press and hold button ▲ to raise to a pre-defined sitting position, by a pre-defined sequence of movements.
3. Press and hold button ▼ to lower to a horizontal position.

Height adjustment



1. Press button (F) to select the function.
2. Press and hold button ▲ to raise the bed.
3. Press and hold button ▼ to lower the bed.

Up-and-Out™ position



1. Press and hold button (B) to move to a pre-defined position to get out of the bed easily.

Back section



1. Press button (C) to select the function.
2. Press and hold button ▲ to raise the back section.
3. Press and hold button ▼ to lower the back section.

Leg section



1. Press button (G) to select the function.
2. Press and hold button ▲ to raise the leg section.
3. Press and hold button ▼ to lower the leg section.

Thigh section



1. Press button (H) to select the function.
2. Press and hold button ▲ to raise the thigh section.
3. Press and hold button ▼ to lower the thigh section.

Anti-Trendelenburg function



1. Press button (D) to select the function.
2. Press and hold button ▲ to tilt to a head end up position.
3. Press and hold button ▼ to tilt back to a horizontal position.

Trendelenburg / Anti-Trendelenburg function



WARNING!

Risk of fatal injury

Tilting with head end down can have fatal effect on users who are sensitive for increased blood pressure in the upper part of the body.

- Only medically trained persons must operate the Trendelenburg function.
- Always perform a medical evaluation, before tilting the bed with the user's lower extremities positioned higher than the heart.



1. Press button (E) to select the function.
2. Press and hold button ▲ to tilt to a head end up position (Anti-Trendelenburg).
3. Press and hold button ▼ to tilt to a head end down position (Trendelenburg).



We recommend locking this function on the hand control, unless it is in use by a healthcare professional.


Under bed light (optional)



1. Press button (I) to switch on or off the under bed light.

4.4.1 Locking function

To prevent the usage of certain functions, each function button can be locked individually.

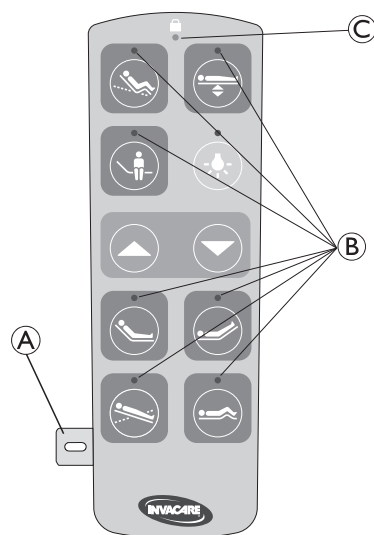


1. Insert the magnetic key (A) in the slot on the side of the hand control.

The blue LED's (B) indicate the locking status of the corresponding function:


- LED on = function is unlocked
- LED off = function is locked

2. To lock or unlock functions, press their corresponding button.
3. Remove the magnetic key (A).

Without the magnetic key (A) inserted, the LED's (B) are off, unless a function is selected. Locked functions can't be selected. When pressing a button of a locked function, the orange LED (C) is blinking to indicate that this function is locked.

 The locking configuration of the hand control remains stored even if the bed is without power supply.

4.5 Hand Control with LCD (optional)

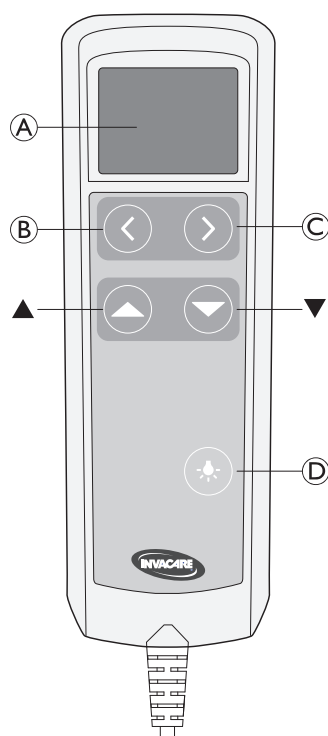

The hand control is equipped with an LCD display (A), navigation buttons (B and C) to select the electrical functions and up and down buttons (▲ and ▼) to control the selected function.

The hand control switches to a stand-by mode when not in use. Touching any button switches the hand control on again.

ErgoMove™



1. Press button (B) or (C) until the symbol for the ErgoMove™ is displayed.
2. Press and hold button ▲ to raise to a pre-defined sitting position, by a pre-defined sequence of movements.
3. Press and hold button ▼ to lower to a horizontal position.

Height adjustment



1. Press button (B) or (C) until the symbol for height adjustment is displayed.
2. Press and hold button ▲ to raise the bed.
3. Press and hold button ▼ to lower the bed.

Up-and-Out™ position



1. Press button (B) or (C) until the symbol for the position is displayed.
2. Press and hold button ▲ or ▼ to move to a pre-defined position to get out of the bed easily.

Back section



1. Press button (B) or (C) until the symbol for head section is displayed.
2. Press and hold button ▲ to raise the back section.
3. Press and hold button ▼ to lower the back section.

Leg section



1. Press button (B) or (C) until the symbol for leg section is displayed.
2. Press and hold button ▲ to raise the leg section.
3. Press and hold button ▼ to lower the leg section.

Thigh section



1. Press button (B) or (C) until the symbol for thigh section is displayed.
2. Press and hold button ▲ to raise the thigh section.
3. Press and hold button ▼ to lower the thigh section.

Anti-Trendelenburg function (optional)

If the bed is equipped with Anti-Trendelenburg function only, tilting with the foot end up is not possible.



1. Press button (B) or (C) until the symbol for the Anti-Trendelenburg function is displayed.

2. Press and hold button ▲ to tilt to a head end up position.
3. Press and hold button ▼ to tilt back to a horizontal position.

Trendelenburg / Anti-Trendelenburg function (optional)



WARNING!

Risk of fatal injury

Tilting with head end down can have fatal effect on users who are sensitive for increased blood pressure in the upper part of the body.

- Only medically trained persons must operate the Trendelenburg function.
- Always perform a medical evaluation, before tilting the bed with the user's lower extremities positioned higher than the heart.



1. Press button B or C until the symbol for the Trendelenburg / Anti-Trendelenburg function is displayed.
2. Press and hold button ▲ to tilt to a head end up position (Anti-Trendelenburg).
3. Press and hold button ▼ to tilt to a head end down position (Trendelenburg).



We recommend locking this function on the hand control, unless it is in use by a healthcare professional.


Under bed light



1. Press button D to switch on or off the under bed light.

4.5.1 Locking function

To prevent the usage of certain functions, each function button can be locked individually.

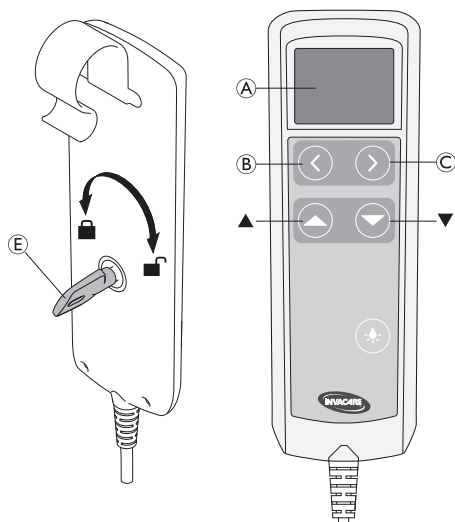

1. Turn the safety key E to the unlocked position.
2. Push button B or C until the respective function is displayed.
The function symbols are accompanied with either a green unlock symbol or a red lock symbol.
3. Press button ▲ to lock the function.
Press button ▼ to unlock the function.
4. Turn the safety key E to the locked position.



The locking configuration of the hand control remains stored even if the bed is without power supply.


4.5.2 Storing preferred positions

The RememberMe™ function allows to memorize a preferred sitting position for ErgoMove™ and a preferred Up-and-Out™ position, additionally to the pre-defined positions.

When a position is memorized, the bed can be moved to this position following the movement to the pre-defined position.



Memorized positions are only available with an LCD hand control connected to the bed.

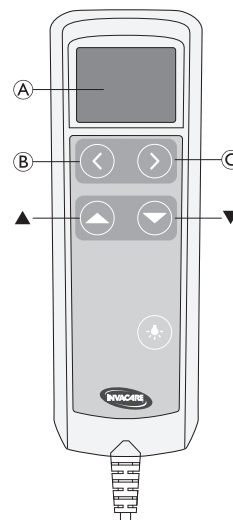

Memorizing sitting position for ErgoMove™

1. Move the bed into the preferred sitting position.
2. Press button B or C until the symbol for the ErgoMove™ is displayed.
3. Press and hold button ▲ and ▼ simultaneously for three seconds.



A storage symbol and a check mark will appear below the ErgoMove™ symbol in the display A to indicate that the position was successfully memorized. The check mark will only be displayed for 1 second.

Memorizing Up-and-Out™ position

1. Move the bed into the preferred position, using the back section and height adjustment.
2. Press button B or C until the symbol for the Up-and-Out™ position is displayed.
3. Press and hold button ▲ and ▼ simultaneously for three seconds.



A storage symbol and a check mark will appear below the Up-and-Out™ symbol in the display A to indicate that the position was successfully memorized. The check mark will only be displayed for 1 second.

Move to the memorized sitting position with ErgoMove™

1. Press button **B** or **C** until the symbol for the ErgoMove™ is displayed.

A storage symbol below the ErgoMove™ symbol in the display **A** indicates that a memorized position is available.



2. Press and hold button **▲** to move to the pre-defined sitting position.
3. Press again and hold button **▲** to move to the memorized sitting position.

Move to the memorized Up-and-Out™ position

1. Press button **B** or **C** until the symbol for the Up-and-Out™ position is displayed.

A storage symbol below the Up-and-Out™ symbol in the display **A** indicates that a memorized position is available.



2. Press and hold button **▲** to move to the memorized position.


Clearing a memorized position

1. Press button **B** or **C** until the symbol for the ErgoMove™ or the Up-and-Out™ position is displayed.
2. Press and hold button **B** or **C** simultaneously for three seconds.

The storage symbol below the ErgoMove™ or Up-and-Out™ symbol in the display **A** disappears indicating that the memorized position was cleared.

4.6 Basic Hand Control

The hand control is equipped with five buttons to operate the electrical adjustment functions of the bed. The buttons on hand control can be locked individually.

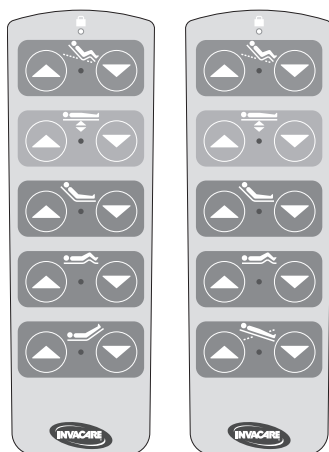

1. Locate the desired function.
2. Press and hold the up (**▲**) button on the left or the down (**▼**) button on the right:

ErgoMove™



- Up (**▲**): Raise to a pre-defined sitting position, by a pre-defined sequence of movements.
- Down (**▼**): Lower to a horizontal position.

Height adjustment



- Up (**▲**): Raise the bed.
- Down (**▼**): Lower the bed.

Back section



- Up (**▲**): Raise the back section.
- Down (**▼**): Lower the back section.

Thigh section



- Up (**▲**): Raise the thigh section.
- Down (**▼**): Lower the thigh section.

Leg section



1. Up (**▲**): Raise the leg section.
2. Down (**▼**): Lower the leg section.

Trendelenburg / Anti-Trendelenburg function (optional)



WARNING!

Risk of fatal injury

Tilting with head end down can have fatal effect on users who are sensitive for increased blood pressure in the upper part of the body.

- Only medically trained persons must operate the Trendelenburg function.
- Always perform a medical evaluation, before tilting the bed with the user's lower extremities positioned higher than the heart.



- Up (**▲**): Tilt to a head end up position (Anti-Trendelenburg).
- Down (**▼**): Tilt to a head end down position (Trendelenburg).




We recommend locking this function on the hand control, unless it is in use by a healthcare professional.

4.6.1 Locking function

(optional)

To prevent the usage of certain functions, each function button can be locked individually.

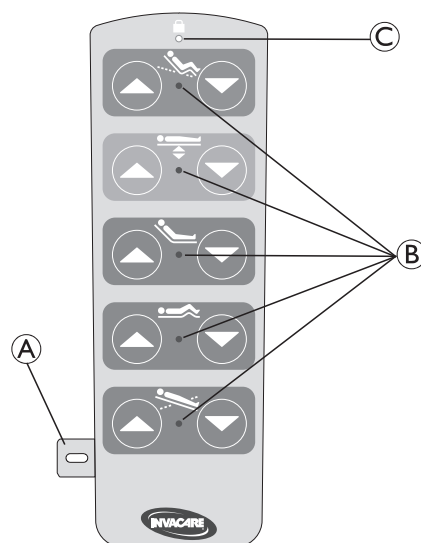

1. Insert the magnetic key ① in the slot on the side of the hand control.

The blue LED's ② on the button indicate the locking status of the corresponding function:

- LED on = function is unlocked
- LED off = function is locked

2. To lock or unlock functions press their corresponding button.
3. Remove the magnetic key ①.

Without the magnetic key ① inserted, the LED's ② are off, unless a function is selected. Locked functions can't be selected. When pressing a button of a locked function, the orange LED ③ is blinking to indicate that this function is locked.



The locking configuration of the hand control remains stored even if the bed is without power supply.

4.7 Operating side rails



WARNING!

Risk of entrapment or suffocation

There's a risk of entrapment or suffocation between mattress support, side rail and bed end.

- Always ensure correct fitting and tightening of the side rails.



WARNING!

Risk of falling

If it has been decided that the user needs side rails:

- Never leave the user unattended in the bed with the side rail down.
- Make sure the side rail is in its highest position and properly locked while leaving the user unattended.



CAUTION!

Risk of Injury

If the side rail is not locked properly it can fall down.

- Pull/push the top bar of the side rail to ensure that the locking system is properly engaged.



CAUTION!


Risk of Injury

There is a risk of entrapment or squeezing while operating the side rail.


- Pay attention to your fingers and body parts of the patient.
- Never force or drop the side rail while handling it.

4.7.1 Operating the Ane side rail

1.

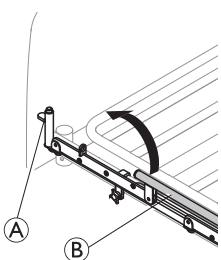


2.

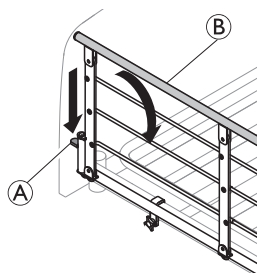


1. **Up:** Lift and pull the top bar ② of the side rail towards the end with the locking mechanism (release button) ①. Ensure, it is locked in place.
2. **Down:** Press the release button ① and push the top bar ② of the side rail away from the locking mechanism.

4.7.2 Operating the Freya / Embla side rail

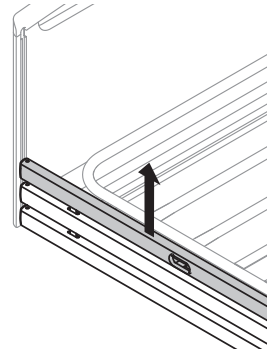


Lock / Upper position

Pull up the top side rail bar, until the locking pin locks with an audible click

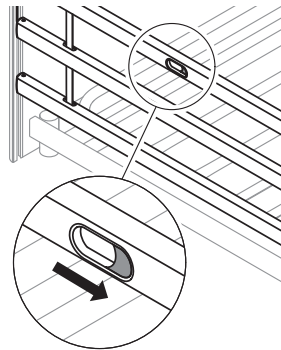

Release

Lift the top side rail bar and press the two locking rings together.

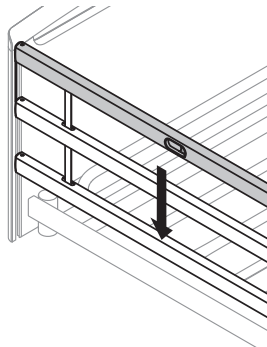

Down / Lower position

Lower the side rail.

Using a side rail height extender

Full length side rails can be supplemented by a height extender. The side rail height extender can be mounted in two height positions and increases the allowed mattress thickness by 6 cm or 10 cm respectively.



WARNING!

Risk of injury

A misapplied side rail height extender could fall off. There's an increased risk for the user to fall out of the bed.

- Make sure that the side rail height extender is properly attached to the side rail.
- Make sure to use the correct height extender for your side rail. The material of the height extender bar must match to the material of the side rail bars.

4.7.3 Operating the Torill side rail



WARNING!
Risk of falling

The middle position of the side rail does not provide protection against falling out of the bed.

- Only the highest position is intended to reduce the risk for the end-user of unintentionally falling out of the bed.
- The middle position must only be used to provide support when the end-user is moving in or out of the bed.


Raising the side rail



NOTICE!


Do not push the release buttons when moving the side rail up.

1.

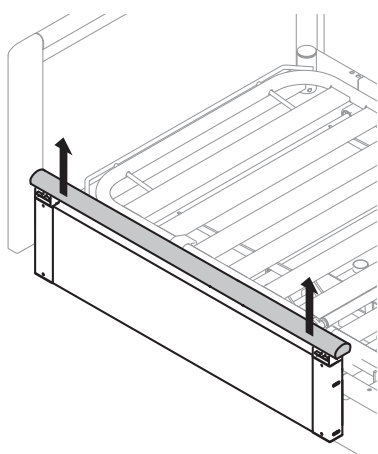

Pull the top bar upwards with both hands near the left and right end.

2.

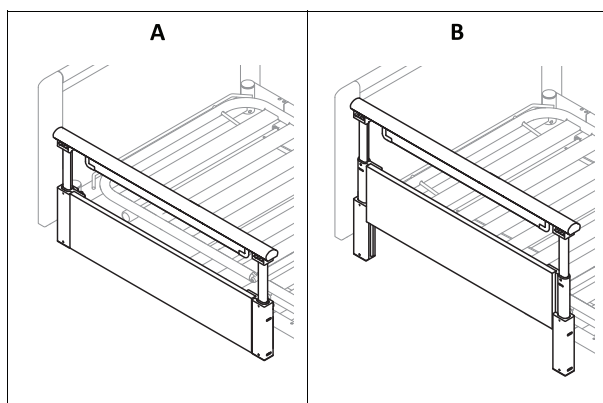

To raise the side rail to the middle position (A), stop pulling upwards after a first audible click.

To raise the side rail to the high position (B), pull upwards to the upper stop.


3. Push the top bar downwards to ensure it is properly locked.



If the side rail does not lock properly, perform the following steps to restore proper function.

- Pull up the top bar upwards to its stop, without pushing the release buttons.
- Lower the side rail to its lowest position as described below.

Lowering the side rail

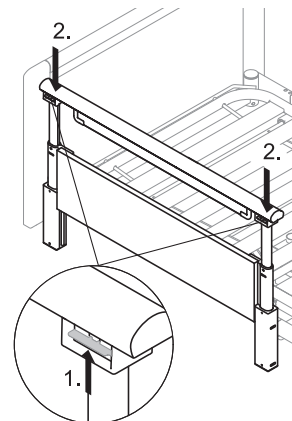

1. Push up and hold the two release buttons.

2. To lower the side rail to the middle position, push the top bar downwards and let go the two release buttons after passing the upper locking position.

To lower the side rail to the lowest position, push the top bar downwards to the lower stop, without releasing the two release buttons.

4.8 Lifting pole




WARNING!
Risk of injury

The bed can tip if the handle is used, while the lifting pole is turned away from the bed.

- The lifting pole always has to be positioned with the handle hanging over the bed area.
- Do not exceed the maximum load of the lifting pole; 80 kg.

4.8.1 Place the lifting pole

The lifting pole can be placed either on the left or the right side of the head end of the bed.

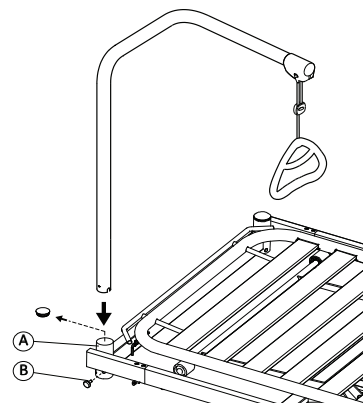

1. Remove the plastic plug from the lifting pole tube (A) at the head end of the bed.


- Insert the lifting pole into the lifting pole tube and fix it with the finger screw ⑥.



The swivel lifting pole must not be fixed with a finger screw, so that it can swing away to the side of the bed.

4.8.2 Adjusting the handle height

The handle height should always be adjusted to the user's need.

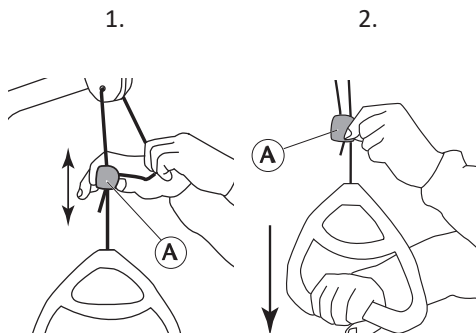

- Hold the cord out while sliding the plastic cord lock ① upwards or downwards until the handle reaches the preferred height.
- Lock the handle by pressing back the cord in the lock ① and pull the handle downwards.



NOTICE!

After adjusting the handle height:

- Check that the two cords above the cord lock are parallel and inside the cord lock.
- Make sure the cord is properly locked by pulling the handle hard.

4.9 Emergency positions

In case of a medical emergency, it might be required to bring all or certain mattress support sections into a flattened position - e.g. a flattened back section for cardiopulmonary resuscitation (CPR).

To bring a mattress support section into a flattened position,

- use the corresponding function on the hand control
OR
- in case of power failure or urgent need for action, use the manual emergency release and lower the mattress support section.

4.9.1 Emergency release of a mattress support section

In case of a power- or motor failure, an emergency release of the back, thigh or leg section could be necessary. An emergency release of the height adjustment is NOT possible.



CAUTION!

Risk of injury


- A minimum of two persons is required for an emergency release of a mattress support section.
- When releasing a mattress support section, it might lower fast. Do not reach under the mattress support while lowering it.



NOTICE!

- Before an emergency release of the mattress support, remove the plug from the mains socket.

- Both persons hold the mattress section.
- One of them locates the motor in question and pulls out the safety pin.

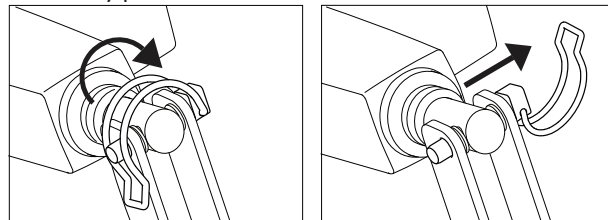

- Both persons slowly lower the mattress section until it is completely down.

4.10 Transportation and Storage

The bed can be transported or stored in assembled or disassembled state.

See the service manual for detailed instructions on how to disassemble the bed.

To transport or store the bed in its assembled state:

- Disconnect the power cable from the mains socket.
- Use the hook near the mains plug to attach the power cable to the bed (e.g. on bed ends or frame) so it is kept clear of the floor and damage during transport or storage is prevented.

5 Maintenance

5.1 General maintenance information


CAUTION!
Risk of injury or damage

- Do not perform any maintenance or service procedures while the product is in use.

Follow the maintenance procedures described in this manual to keep your product in continuous operation.

Further maintenance and inspection procedures, to be performed by a qualified technician, are described in the service manual of this product. Service manuals can be obtained from Invacare.

5.1.1 Service interval

We strongly recommend a full service to be performed at least every second year and a yearly check of the actuators.

Contact your Invacare provider or in-house service department for service.

5.2 Daily inspections


WARNING!
Risk of injury or damage

Damaged or worn parts can affect the safety of the product.

- The product should be checked each time it is used.
- Do not use the product if damage is found or you question the safety of any part of the product. Contact your Invacare provider immediately and ensure the product is not used until repairs are made.

Daily Inspection Checklist

- ☐ Visually inspect the bed. Check all parts for external damage or wear.
- ☐ Check all hardware, attachment points, stressed parts and any pivot points for signs of wear, cracking, fraying, deformation or deterioration.
- ☐ Check the hand control and all actuators for smooth operation.

5.3 Cleaning and Disinfection

5.3.1 General Safety Information


CAUTION!
Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.


CAUTION!
Risk of Electric Shock and Product Damage

- Switch off the device and disconnect from mains, if applicable.
- When cleaning electronic components consider their protection class regarding water ingress.
- Make sure that no water splashes to the plug or the wall outlet.
- Do not touch the power socket with wet hands.


NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking into use again.



For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

5.3.2 Cleaning Intervals


NOTICE!

Regular cleaning and disinfection enhance smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

5.3.3 Cleaning Instructions



NOTICE!

Only certain components of the product tolerate cleaning in automatic washing systems or with high pressure cleaning equipment.

- For detailed information on cleaning in automatic washing systems or with high pressure cleaning equipment see the Service Manual of the product.
- All other components must be cleaned by hand.

Cleaning by Hand

All components (excluding removable textiles)

- Cleaning agent: We recommend using a mild detergent with neutral or near neutral pH (5-9).
 Commercially available consumer products like hand dishwashing liquid or all-purpose cleaners can be used. Read the instructions on the label of your cleaning agent and use it at the indicated concentration.
 - Max. water temperature: 40 °C
1. Wipe surfaces thoroughly with a soft cloth slightly moistened with detergent solution until all visible soil is removed.
 2. Remove all detergent residue with a clean damp cloth.
 3. Wipe dry using a clean soft cloth.


Removable textiles

(including upholstery and mattresses)

- See attached label on each product.

5.3.4 Disinfection Instructions

In Domestic Care

- Disinfectant: We recommend using an alcohol-based surface disinfectant (with 70-90% alcohol).
 Read the instructions on the label of your disinfectant. It provides information on the activity spectrum (bacteria, fungi and/or viruses), material compatibility and the correct exposure time.
1. Ensure the surfaces are cleaned before disinfection.
 2. Moisten a soft cloth and wipe-disinfect all accessible surfaces and keep them moistened for the exposure time stated on the label of the disinfectant.
 3. Allow the product to air-dry.

In Institutional Care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

6 After Use

6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, page 18* and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

7 Troubleshooting

7.1 Troubleshooting


Symptom	Possible cause	Remedy
Bed section does not move	End of stroke reached	Operate opposite button
Full bed: No electric function does work	Bed not plugged in	Plug in mains cable
	Hand control not connected	Ensure hand control correctly connected to control unit
	Hand control defective	Call dealer / technician to replace hand control
	Control unit defective	Call dealer / technician to replace control unit
	Cables entangled or pinched	Call dealer / technician to replace cables
Side rail does not open or close	Locking mechanism defective	Call dealer / technician to repair side rail

8 Technical Data

8.1 Materials

Mattress Platform	Steel (powder-coated)
Base Frame and Shear arms	Steel (powder-coated)
Bed Ends	Wood and/or MDF (Zinc-plated fixation brackets)
Side Rails	Steel (powder-coated), Wood or Aluminium
Lifting Pole	Steel (powder-coated), PP and POM
Actuators housing, Hand control, Control box, Castors and other Plastic parts	Material according to marking (PA, PP, PE, ABS)
Bolts and nuts	Steel (Zinc-plated)


8.2 Dimensions bed

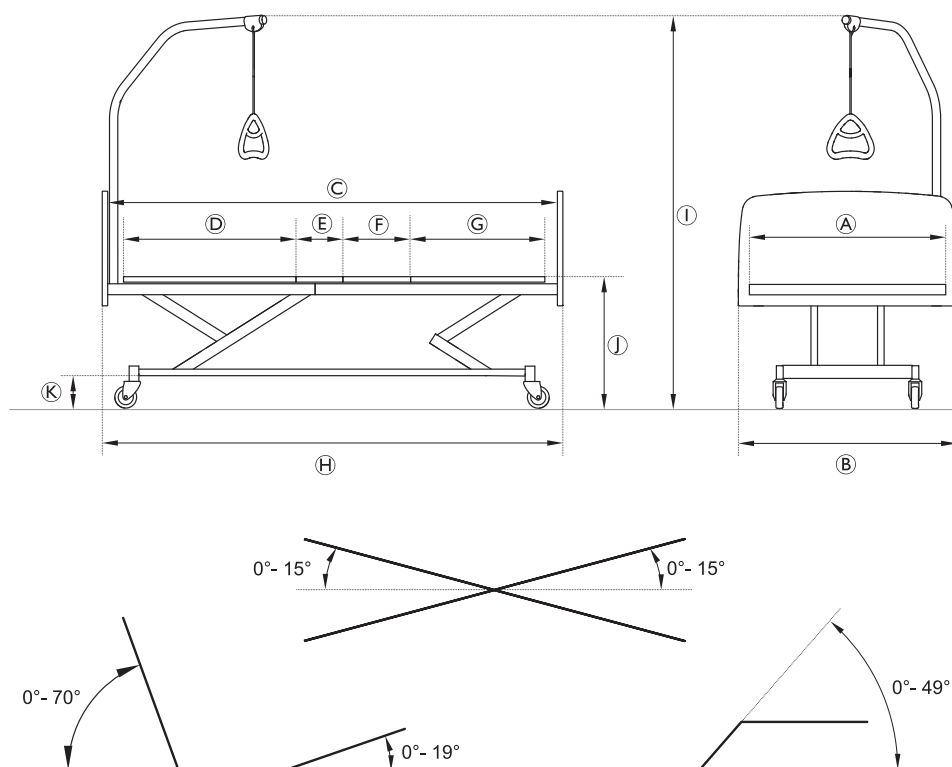



Dimensions [cm]	NordBed Ultra	NordBed Optimo		
Mattress support width ①	90	90	105	120
Total width ②	100	100	115	130
Mattress support length ③	200			
Back section length ④	80			
Seat section length ⑤	17			
Tight section length ⑥	30			
Leg section length ⑦	65			
Total length ⑧	213			
Total height (with lifting pole) ⑨	160 - 207			
Mattress support to floor height ⑩ – standard configuration	25 - 85	32 - 81		32 - 81

Dimensions [cm]	NordBed Ultra	NordBed Optimo	
Mattress support to floor height ① – low configuration	—	28 - 77	—
Base to floor height ②	15		

! NOTICE!
The low configuration of the bed does not meet the standard requirements for the upper value of the mattress support to floor height.

 Angles states with tolerances of $\pm 3^\circ$. All dimensions are stated without tolerances for a non-extended bed with castors of 10 cm diameter. Outer dimensions are stated for a bed with Nora bed end and Freya side rail.

8.3 Maximum loads

		NordBed Ultra	NordBed Optimo 90 cm wide	NordBed Optimo 105 cm wide	NordBed Optimo 120 cm wide
	Max. safe working load (including accessories)	235 kg			285 kg
	Max. user weight (provided that the weight of the mattress and the accessories do not exceed 35 kg)	200 kg			250 kg

8.4 Weights

Bed and heaviest parts

	NordBed Ultra	NordBed Optimo	NordBed Optimo Wide	
	width 90 cm	width 90 cm	width 105 cm	width 120 cm
Complete bed (excl. accessories)	105 kg	100 kg	105 kg	130 kg
Mattress support, upper half	24 kg	24 kg	26 kg	30 kg
Mattress support, lower half	20 kg	20 kg	23 kg	30 kg
Base	61 kg	—	—	70 kg
Lower base	—	22 kg	22 kg	—
Scissor upper half	—	18 kg	18 kg	—
Scissor lower half	—	16 kg	16 kg	—

Side rails

Embla	4.0 kg/pc
Embla extended	6.0 kg/pc
Freya	6.1 kg/pc
Freya +10 cm	6.4 kg/pc
Freya +20 cm	6.7 kg/pc
Ane	8.8 kg/pc
Ane (83/85)	8.8 kg/pc
Ane High	10.9 kg/pc
Ane High (83/85)	10.9 kg/pc
Torill Short	6.2 kg/pc
Torill Long	7.7 kg/pc
Height extender for Embla	2.5 kg/pc
Height extender for Freya	3.0 kg/pc

Bed ends

	Mattress support width 90 cm	Mattress support width 105 cm	Mattress support width 120 cm
Una	11.3 kg/pc	—	—
Footpanel for Una	5.0 kg/pc	—	—
Nora	7.3 kg/pc	8.1 kg/pc	8.8 kg/pc
Astrid	10.5 kg/pc	—	—
Nanna	10.6 kg/pc; 7.6 kg/pc	—	—
Inga and Runa	7.5 kg/pc	8.3 kg/pc	9.1 kg/pc

Wooden side panel

	Panel length 200 cm	Panel length 210 cm	Panel length 220 cm
For Una, Nora and Astrid	5.7 kg/pc	5.9 kg/pc	6.1 kg/pc
For Nanna	5.7 kg/pc	5.9 kg/pc	6.1 kg/pc
For Inga and Runa	4.4 kg/pc	4.6 kg/pc	4.8 kg/pc

8.5 Dimensions side rails

Side rail	Type	Measures	Information
Embla	Full length aluminium side rail	206 cm x 37 cm	Mounted on the bed end guideways. Release buttons in the middle.
Embla extended	Full length aluminium side rail with telescopic extension	206 - 226 cm x 35 cm	Mounted on the bed end guideways. Release buttons in the middle.
Freya	Full length wood side rail	206 cm x 35 cm	Mounted on the bed end guideways. Release buttons in the middle.
Freya +10 cm	Full length wood side rail	216 cm x 35 cm	Mounted on the bed end guideways. Release buttons in the middle.
Freya +20 cm	Full length wood side rail	226 cm x 35 cm	Mounted on the bed end guideways. Release buttons in the middle.

Side rail	Type	Measures	Information
Ane	3/4 length collapsible steel side rail	160 cm x 46 cm	Mounted on the side tube. Release button in the head section.
Ane High	3/4 length collapsible steel side rail	160 cm x 55 cm	Mounted on the side tube. Release button in the head section.
Torill Short	Split telescopic side rail	76 cm x 56 cm	Mounted on the side tube. Release buttons in both ends.
Torill Long	Split telescopic side rail	98 cm x 56 cm	Mounted on the side tube. Release buttons in both ends.

8.6 Allowed mattress sizes

Allowed mattress height and length depending on the side rail in use

	Allowed mattress sizes		
Side rails	Max height [cm]	Min height [cm]	Min length [cm]
Ane / Ane (83/85)	18	10	200
Ane High / Ane High (83/85)	28	18	200
Freya	17	10	200
Embla	17	10	200
Torill	19	14	200

Allowed mattress width

Side rails	Bed width	Allowed mattress width
Ane (83/85) / Ane High (83/85)	90 cm	83 - 85 cm
Ane / Ane High	90 cm	88 - 90 cm
Freya	105 cm	103 - 105 cm
Embla		
Torill	120 cm	118 - 120 cm

8.7 Electrical system

Voltage supply	Uin 230 VAC, 50 Hz (AC = Alternating current)
Maximum current input	Iin max.4 Ampere
Intermittent(periodic motor operation)	10%, max. 2 min ON / 18 min OFF
Insulation class	CLASS II equipment
Type B Applied Part	Applied Part complying with the specified requirements for protection against electrical shock according to IEC60601-1.
Sound level	55 dB (A)
Degree of protection	<p>The control unit, external power supply, motors and hand controls are protected according to IPx6 or IPx6W (depending on configuration). See bed identification label and label on each electric device for correct IP class. The lowest IP-classification decides the overall classification of the bed.</p> <p>IPx6 - The system is protected against powerful water jets projected from any direction.</p> <p>IPx6w - The system is protected against powerful water jets projected from any direction and additional weather conditions to be machine washable according with IEC60601-2-52.</p>



The bed has no isolator (main switch). If the bed needs to be electrically disconnected, it has to be unplugged from the mains socket.

8.8 Environmental conditions

	Storage and Transportation	Operation
Temperature	-10 °C to +50 °C	+5 °C to +40 °C
Relative humidity	20% to 90% at 30 °C - not condensing	
Atmospheric pressure	860 hPa to 1060 hPa	



Allow the product to reach operation temperature before usage:

- Warming up from minimum storage temperature might take up to 24 hours.
- Cooling down from maximum storage temperature might take up to 24 hours.

9 Electromagnetic compatibility (EMC)

9.1 General EMC information

Medical Electrical Equipment needs to be installed and used according to the EMC information in this manual.

This product has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2 for Class B equipment.

Portable and mobile RF communications equipment can affect the operation of this product.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standard. To determine if the emission from this product is causing the interference, run and stop running this product. If the interference with the other device operation stops, then this product is causing the interference. In such rare cases, interference may be reduced or corrected by the following:

- Reposition, relocate, or increase the separation between the devices.

9.2 Electromagnetic emission

Guidance and manufacturer's declaration


This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group I	This product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This product is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Complies	

9.3 Electromagnetic Immunity

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines; 100 kHz repetition frequency ± 1 kV for input / output lines; 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line to line ± 2 kV line to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 0% U_T for 0,5 cycle at 45° steps 0% U_T for 1 cycles 70% U_T for 25 / 30 cycles < 5% U_T for 250 / 300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of this product requires continued operation during power mains interruptions, it is recommended that the product is powered from an uninterruptible power supply or a battery. U_T is the a. c. mains voltage prior to application of the test level.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 V150 kHz to 80 Mhz	Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which this product is used exceeds the applicable RF compliance level above, this product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating this product.
Radiated RF IEC 61000-4-3	6 Vin ISM & amateur radio bands 10 V/m80 Mhz to 2,7 GHz 385 MHz - 5785 MHz test specifications for immunity to RF wireless communication equipment refer to table 9 of IEC 60601-1-2	Interference may occur in the vicinity of equipment marked with the following symbol:  Portable and mobile RF communications equipment should be used no closer than 30 cm to any part of this product including cables.



These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

9.3.1 EMC test specifications

IEC 60601-1-2 — Table 9

Test Frequency (MHz)	Band a) (MHz)	Service a)	Modulation	Immunity test level (V/m)
385	380 - 390	TETRA 400	Pulse modulation ^{b)} 18 Hz	27
450	430 - 470	GMRS 460, FRS 460	FM ^{c)} ± 5 kHz deviation 1 kHz sine	28
710 745 788	704 - 787	LTE Band 13, 17	Pulse modulation ^{b)} 217 Hz	9
810 870 930	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation ^{b)} 18 Hz	28
1720 1845 1970	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation ^{b)} 217 Hz	28
2450	2400 - 2570	Bluetooth, WLAN, 802.11, b/g/n, RFID, 2450, LTE, Band 7	Pulse modulation ^{b)} 217 Hz	28
5240 5500 5785	5100 - 5800	WLAN 802.11a/n	Pulse modulation ^{b)} 217 Hz	9



If necessary to achieve the immunity test level, the distance between the transmitting antenna and the Medical Electrical Equipment or System may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

a) For some services, only the uplink frequencies are included.

b) The carrier shall be modulated using a 50 % duty cycle square wave signal.

c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

**Belgium & Luxemburg:**

Invacare nv
Autobaan 22
B-8210 Loppem
Tel: (32) (0)50 83 10 10
Fax: (32) (0)50 83 10 11
marketingbelgium@invacare.com
www.invacare.be

Danmark:

Invacare A/S
Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
Fax: (45) (0)36 90 00 01
denmark@invacare.com
www.invacare.dk

France:

Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
contactfr@invacare.com
www.invacare.fr

Italia:

Invacare Mecc San s.r.l.,
Via Marco Corner, 19
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59
servizioclienti@invacare.com
www.invacare.it

Nederland:

Invacare BV
Galvanistraat 14-3
NL-6716 AE Ede
Tel: (31) (0)318 695 757
nederland@invacare.com
www.invacare.nl

Norge:

Besøksadresse:
(Office addresses)
Invacare AS
Brynsveien 16
0667 Oslo
Tel: (47) 22 57 95 00
norway@invacare.com
www.invacare.no

Norge:

Vareleveringsadresse:
(Storage / Technical dep)
Østensjøveien 19
0661 Oslo
eknisk@invacare.com
www.invacare.no

Portugal:

Invacare Portugal II, Lda.
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 193 360
portugal@invacare.com
www.invacare.pt

Suomi:

Camp Mobility
Patamäenkatu 5, 33900 Tampere
Puhelin 09-35076310
info@campmobility.fi
www.campmobility.fi

Sverige:

Invacare AB
Fagerstagatan 9
S-163 53 Spånga
Tel: (46) (0)8 761 70 90
Fax: (46) (0)8 761 81 08
sweden@invacare.com
www.invacare.se



Invacare Rea AB
Växjövägen 303
343 71 Diö
Sverige

