
Invacare® Ane

Invacare® Ane, Invacare® Ane High

en	Side rail for Invacare® NordBed™ User Manual	1
da	Sengehest til Invacare® NordBed™ Brugsanvisning.....	2
it	Sponda laterale per Invacare® NordBed™ Manuale d'uso	2
nl	Zijhek voor Invacare® NordBed™ Gebruiksaanwijzing	4
no	Sengegrind til Invacare® NordBed™ Bruksanvisning.....	5
sv	Sänggrind för Invacare® NordBed™ Bruksanvisning.....	5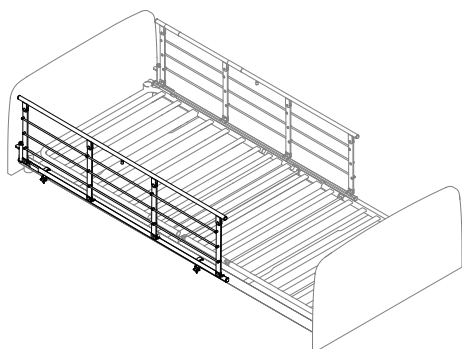

1 Introduction

This manual belongs to an Invacare accessory and is an addition to the manuals of the related Invacare product.

To ensure safety when using this accessory, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.

2 Installing side rails

WARNING!

Risk of personal damage

There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.

- Pay attention to your fingers.
- Follow instructions carefully.
- After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

Removable 3/4 length side rails

WARNING!

Risk of entrapment


There's a risk of entrapment or suffocation between mattress support, side rail and bed end.

- When using removable side rails, always make sure that the distance between the bed end and the side rail's handle upper edge is less than 6 cm in the head end and more than 32 cm in the foot end of the bed.

2.1 Installing the Ane side rail


Mounting

1.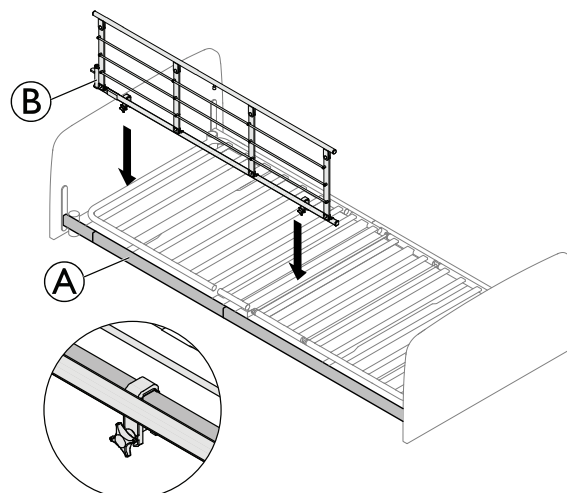

Place the mounting clamps of the side rail over the square tube (A), with the release system (B) at the head end of the bed.

2.

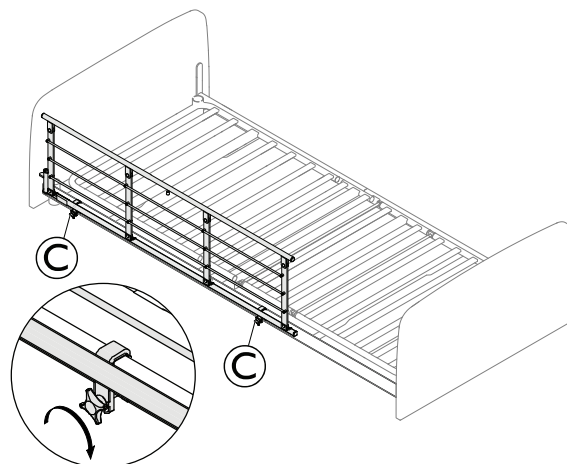This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®


Tighten the finger screws © to attach the side rail on the square tube (A) and make sure it is steady and fastened.

Dismounting


1. Loosen the finger screws © and remove the side rail.

3 Operating the Ane side rail

1.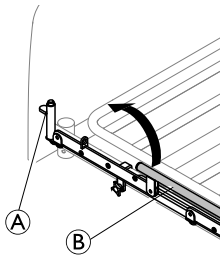

2.

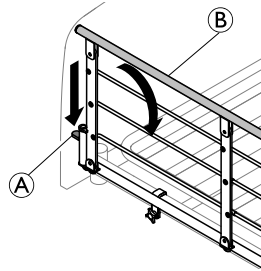1. **Up:** Lift and pull the top bar (B) of the side rail towards the end with the locking mechanism (release button) (A). Ensure, it is locked in place.
2. **Down:** Press the release button (A) and push the top bar (B) of the side rail away from the locking mechanism.

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Invacare AB
Invacare Limited
Invacare Rea AB