## Wireless Hand Control

Invacare® NordBed™ Ultra, Invacare® NordBed™ Optimo,  
Invacare® NordBed™ Optimo Wide

en	<b>Wireless Hand Control User Manual</b> .....	2
da	<b>Trådløs håndbetjening Brugsanvisning</b> .....	4
fi	<b>Langaton käsiohjain Käyttöohje</b> .....	6
it	<b>Pulsantiera wireless Manuale d'uso</b> .....	8
nl	<b>Draadloze handbediening Gebruiksaanwijzing</b> .....	11
no	<b>Trådløs håndkontroll Bruksanvisning</b> .....	13
sv	<b>Trådlös manöverdosa Bruksanvisning</b> .....	16

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.®**

## 1 Introduction

This manual belongs to an Invacare accessory and is an addition to the manuals of the related Invacare product.

To ensure safety when using this accessory, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.

## 2 General information

The wireless hand control is a secondary hand control, to be used in combination with a tethered hand control. A tethered hand control is required for more advanced functions, like locking or unlocking functions and resetting the system.

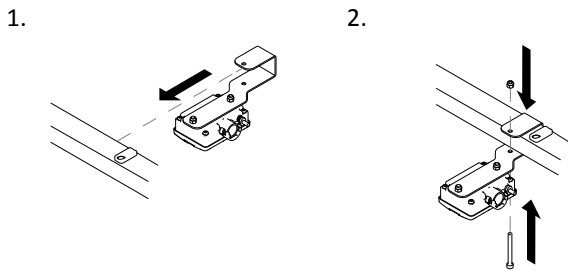
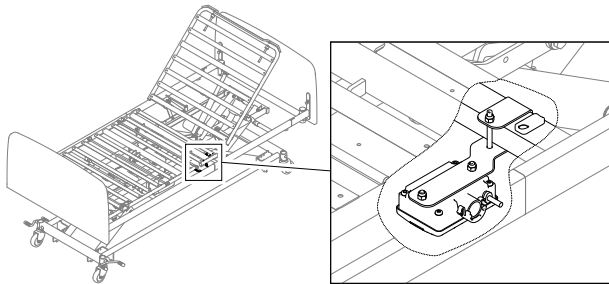
The system consists of:

- 1x Wireless hand control
- 1x Receiver

## 3 Installing the Wireless Hand Control



### Mounting the receiver

-  1x 4 mm Allen key, 1x 8 mm Wrench


1. Place the mounting bracket of the receiver over the square tube of the mattress support platform.
2. Fix the receiver with the bolt and nut.


### Wiring

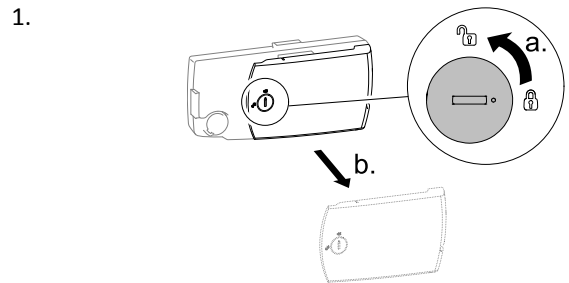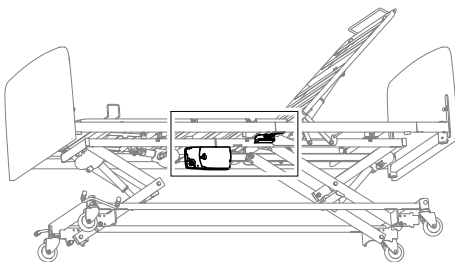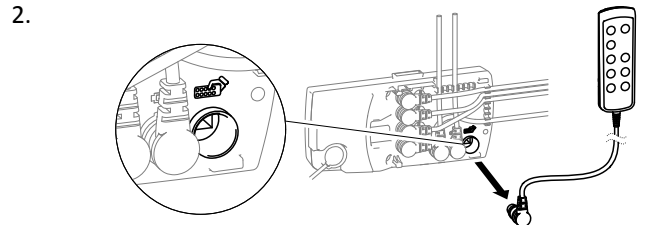
On the control unit turn the lock 90 degrees counter-clockwise to open the lid and remove it.

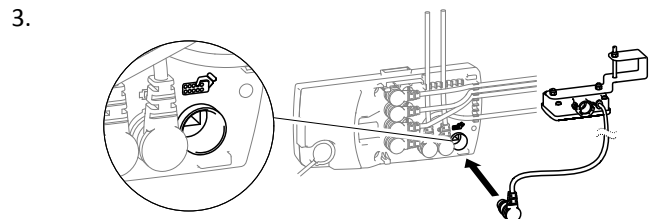
Disconnect the cable of the tethered hand control from the control unit.

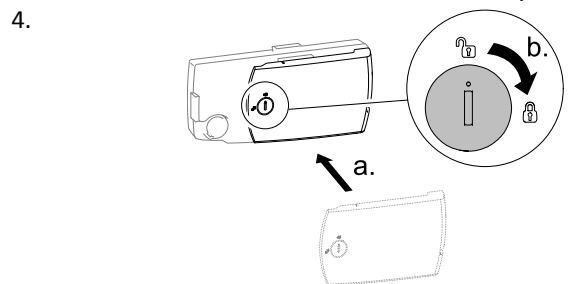Connect the cable from the receiver to the control unit in the socket labeled with hand control symbol.

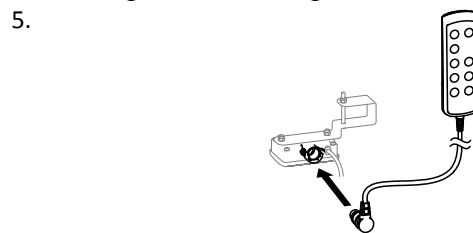Put the lid back on the control unit and close it by turning the lock 90 degrees clockwise.


Connect the cable of the tethered hand control to the receiver.

## 4 Pairing

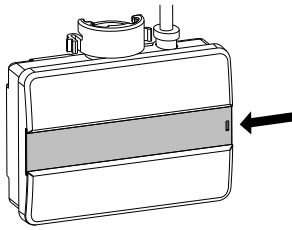(optional)


-  If the wireless hand control and the receiver are delivered as a set, a initial pairing is not required. Pairing is only required for a changed combination of wireless hand control and receiver or if the connection is lost.

1.

Slide open the lid on the receiver.

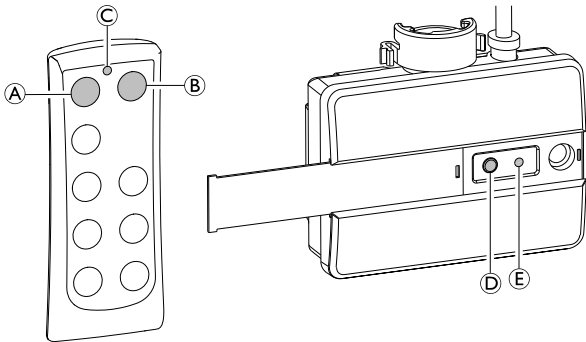2.


a. Push and hold buttons **A** and **B** on the hand control simultaneously for five seconds until the LED **C** starts flashing to indicate that the hand control is in pairing mode.

 The pairing mode will be active for seven second and automatically stop if a pairing is not established within this time frame.


b. While the hand control is in pairing mode, push and hold button **D** on the receiver until the LED **E** is flashing twice to indicate that the pairing was successful.

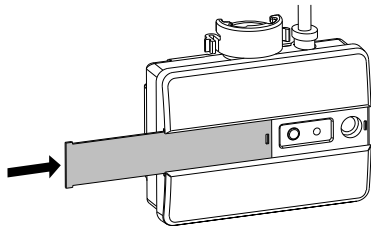3.

Close the lid on the receiver.

## 5 Using the Wireless Hand Control

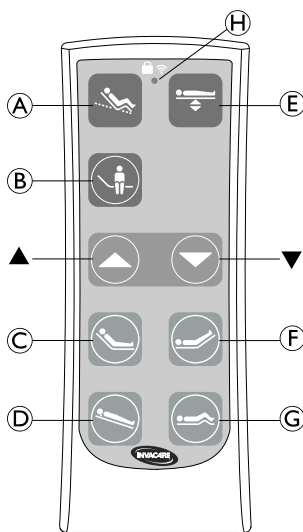The hand control is equipped with buttons for electrical functions (**A** to **G**), up and down buttons (**▲** and **▼**) to control the selected function

When a button is pressed and the hand control is communicating with the receiver the LED **H** will flash in blue.

When a locked function is selected the LED **H** will flash in yellow.

1660535-B

## ErgoMove™

1. Press button **A** to select the function.
2. Press and hold button **▲** to raise to a pre-defined sitting position, by a pre-defined sequence of movements.
3. Press and hold button **▼** to lower to a horizontal position.

## Height adjustment

1. Press button **E** to select the function.
2. Press and hold button **▲** to raise the bed.
3. Press and hold button **▼** to lower the bed.

## Up-and-Out™ position

1. Press and hold button **B** to move to a pre-defined position to get out of the bed easily.

## Back section

1. Press button **C** to select the function.
2. Press and hold button **▲** to raise the back section.
3. Press and hold button **▼** to lower the back section.

## Leg section

1. Press button **F** to select the function.
2. Press and hold button **▲** to raise the leg section.
3. Press and hold button **▼** to lower the leg section.

## Thigh section

1. Press button **G** to select the function.
2. Press and hold button **▲** to raise the thigh section.
3. Press and hold button **▼** to lower the thigh section.

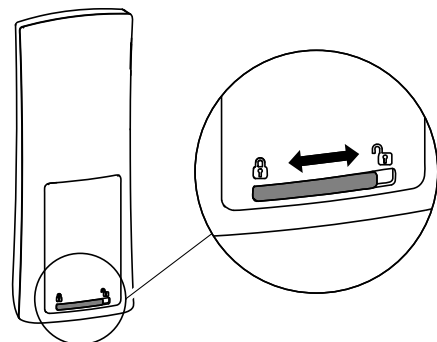
## Anti-Trendelenburg function

1. Press button **D** to select the function.
2. Press and hold button **▲** to tilt to a head end up position.
3. Press and hold button **▼** to tilt back to a horizontal position.

## 6 Exchanging batteries

The wireless hand control is powered by 2 AAA 1.5 V batteries.

To exchange the batteries:


1. Unlock the lid on the back side of the hand control and remove it.
2. Replace the batteries with batteries of the same type.
3. Put the lid back and lock it.

**WARNING!****Environmental Hazard**

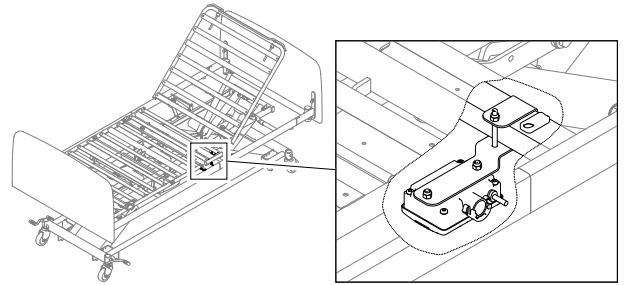
Disposal of batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

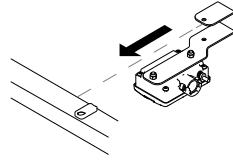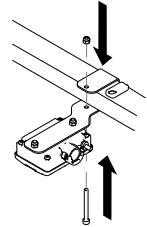
- DO NOT dispose of batteries in normal household waste.
- Batteries MUST be taken to a proper disposal site. The return is required by law and free of charge.
- Do only dispose discharged batteries.
- Cover terminals of lithium batteries prior to disposal.
- For information on the battery type see battery label.

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©2020 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare Limited
Invacare Rea AB