Küschall The KSL Adjustable backrest

Fig 1

74°	78°	82°	86°	90°

Fig 2

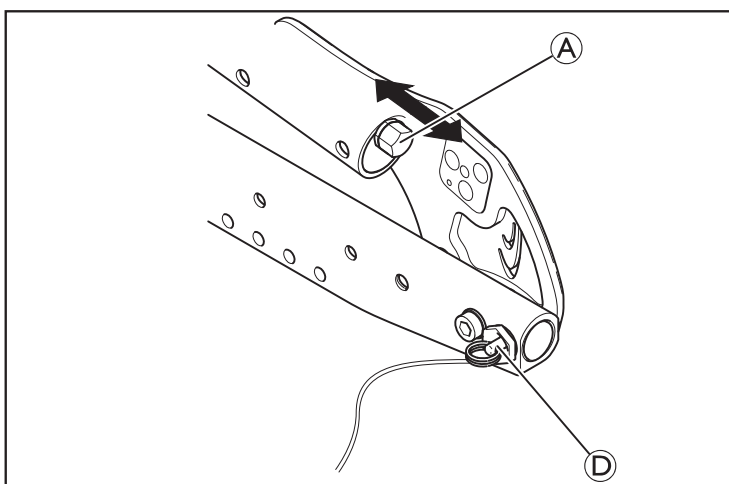
Fig 3


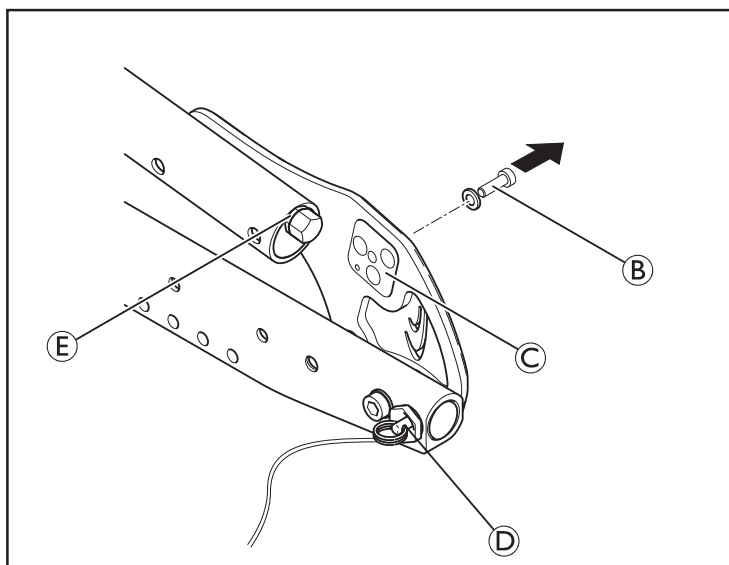
Fig 4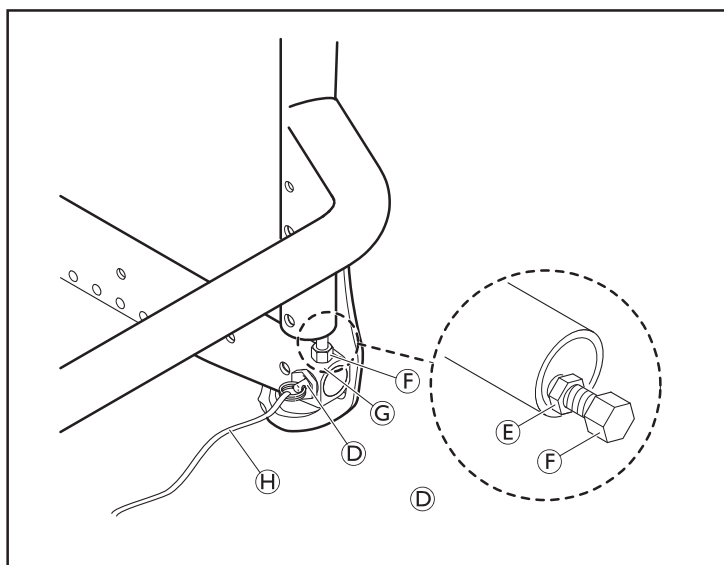

en **Adjusting the backrest angle**
Instruction sheet

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Yes, you can.®

The angle of the backrest can be changed by repositioning the excenter plate in the backrest joint plate. The following angles (measured from the seat) can be set (figure 1).

Adjusting the backrest angle

Allen key (3 mm) / Wrench (10 mm)

1. Fold down the backrest and release the counter nut of the adjustment screw (A) and screw it in completely.
2. Remove the screw (B) on the excenter plate (C). Remove the excenter plate and reinsert it in the required position. The excenter plates must be positioned identically on both sides in the backrest joint.
3. Reinsert the screw (B) and tighten it.
4. Adjust the backrest joint (figure 3).
5. Fold up the backrest and let the pin (A) engage.
6. Press the backrest forwards to ensure the joint cannot move.
7. Unscrew the screw (F) until it touches the seat frame (G).
8. Screw the screw back in by between a ¼ and a ½ turn and counter with the counter nut (E).
9. Make sure that the backrest joint is adjusted correctly on both sides.
 (B) = 4 Nm
 (E) = 7 Nm

Function control:

Sit in the wheelchair and lean back so that the backrest is strained. Upon pulling the release cord (H), the pin (D) must be easily removed on both sides and must fully glide back in upon letting go of the release cord.

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Inbusschlüssel (3 mm)/Schraubenschlüssel (10 mm)

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Invacare AB
Invacare AG
Invacare Limited
Invacare France Operations SAS