

Soft Tilt™



Innovative support and
positioning



Yes, you can.®

The **Invacare Soft Tilt** is a unique and innovative new system designed to automatically reposition patients, as well as supporting care staff with daily moving and handling procedures.

The **Soft Tilt** can be easily installed on to most **Invacare** beds, and will work in conjunction with the beds existing profiling movements and mattress.

- ▶ Automated pressure relief
- ▶ Optimised for single carer working
- ▶ Enhanced patient and carer comfort



Automated pressure relief


The **Soft Tilt** can be programmed to automatically turn patients at various intervals and angles throughout the night; relieving pressure and preventing the development of pressure ulcers. When operating automatically, the wings will never elevate higher than 40° to ensure optimal safety and comfort whilst the patient is sleeping.

Single carer working

The **Soft Tilt** can be operated manually by a carer, making it ideal for moving and handling within single carer working environments. The easy to use hand control allows the carer to elevate the wings to up to 80°, giving full access for sling application, glide sheets and personal care. Furthermore, when used in conjunction with the beds profiling movements, the **Soft Tilt** allows the carer to safely transfer the patient in and out of bed - into a standing position - with minimal effort, protecting the carer from potential back injury.

Enhanced patient and carer comfort

Thanks to the silent, gentle and automated movements of the **Soft Tilt**, patients can enjoy a more peaceful, undisturbed sleep. Moreover, carers benefit from a more comfortable working environment and avoid the risk of back injury, normally caused when moving and handling.

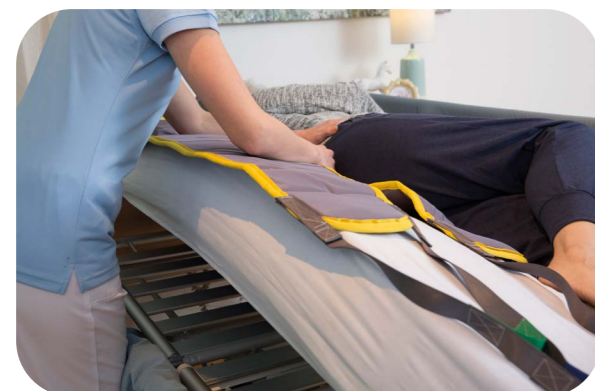

Key features and benefits



▲ Profiling

Works in combination with the beds profiling movements to ensure comfortable positioning of the patient as well as supporting with transfers.



▲ Manual operation

When operated manually with the handset, the wings can be elevated to 80° for moving and handling purposes, such as personal care and sling application.



◀ Programmable control box

The easy to use control box comes with one preset program (auto) and two customisable programs to suit individual user requirements. Once programmed, automated movements can be activated at the touch of a button.



▲ Cradling for comfort

Soft Tilt can be programmed to turn the patient gently from side to side when both wings are elevated.



▲ Intuitive handset

Clear and easy to use handset for manual use. One intuitive button which activates both wings for a comfortable manual repositioning.



▲ Cover

Blue cover with elastic straps helps keep the mattress in place. Machine washable up to 80 degrees.

Invacare range compatible with **Soft Tilt**



▶ Invacare® **SB 755**



NordBed Range:


- ▶ Invacare® **NordBed Optimo**,
- ▶ Invacare® **NordBed Ultra**




▶ Invacare® **SB 910**

Technical data ▼

For more comprehensive pre-sales information about this product, including the product's user manual, please see your local Invacare website.

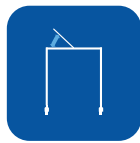

Max angle in manual mode ▼

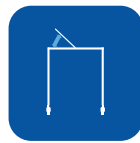

Max angle in automated mode ▼



Time values in automated mode ▼



Max. user weight ▼



Heaviest part of the product ▼



Cover washable ▼

	Max angle in manual mode ▼	Max angle in automated mode ▼	Time values in automated mode ▼	Max. user weight ▼	Heaviest part of the product ▼	Cover washable ▼
Soft Tilt	80°	40°	0 - 180 minutes	116 - 165 kg * SWL 145 - 185 kg *	18 kg	80°

Complies with IEC 60601-2-52

* Depending on which bed the Soft Tilt is mounted on. Please see Soft Tilt user manual for the selected bed.

Frame colour ▼



White

Please notice colors may vary slightly from those displayed above.

NB: The bed must not be used by patients under 12 years of age, or by patients with body size equivalent to an average 12 year old or smaller.

Soft Tilt can only be used in combination with **SB 755**, **NordBed Optimo**, **NordBed Ultra** and **SB 910**.

The **Soft Tilt** is manufactured by GDV Technology on behalf of Invacare.

Tel: +41 61 487 70 70
hqeurope@invacare.com
www.invacare.eu.com

Invacare International GmbH
Neuhofweg 51
4147 Aesch BL
Switzerland



Yes, you can.®

