



Invacare® | **Soft Tilt™**

Innovative support and positioning



The *Invacare Soft Tilt* is a unique and innovative new system designed to automatically reposition patients, as well as supporting care staff with daily moving and handling procedures.

The *Soft Tilt* can be easily installed on to the *Invacare SB® 755* mattress platform, and will work in conjunction with the beds existing profiling movements and mattress.

- Automated pressure relief
- Optimised for single carer working
- Enhanced patient and carer comfort



Automated pressure relief

The *Soft Tilt* can be programmed to automatically turn patients at various intervals and angles throughout the night; relieving pressure and preventing the development of pressure ulcers. When operating automatically, the wings will never elevate higher than 30° to ensure optimal safety and comfort whilst the patient is sleeping.

Single carer working

The *Soft Tilt* can be operated manually by a carer, making it ideal for moving and handling within single carer working environments. The easy to use hand control allows the carer to elevate the wings to up to 60°, giving full access for sling application, glide sheets and personal care. Furthermore, when used in conjunction with the beds profiling movements, the *Soft Tilt* allows the carer to safely transfer the patient in and out of bed - into a standing position - with minimal effort, protecting the carer from potential back injury.

Enhanced patient and carer comfort


Thanks to the silent, gentle and automated movements of the *Soft Tilt*, patients can enjoy a more peaceful, undisturbed sleep. Moreover, carers benefit from a more comfortable working environment and avoid the risk of back injury, normally caused when moving and handling.





Profiling

Works in combination with the beds profiling movements to ensure comfortable positioning of the patient as well as supporting with transfers.

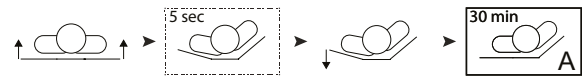

Comfortable, automated pressure relief

When operating automatically, the secondary wing will elevate slightly to offer additional support and comfort for the patient. Once the patient reaches the desired position, the secondary wing will return to a horizontal position.




Manual operation

When operated manually with the handset, the wings can be elevated to 60° for moving and handling purposes, such as personal care and sling application.



Programmable control box

The easy to use control box comes with one preset program (auto) and two customisable programs to suit individual user requirements. Once programmed, automated movements can be activated at the touch of a button.

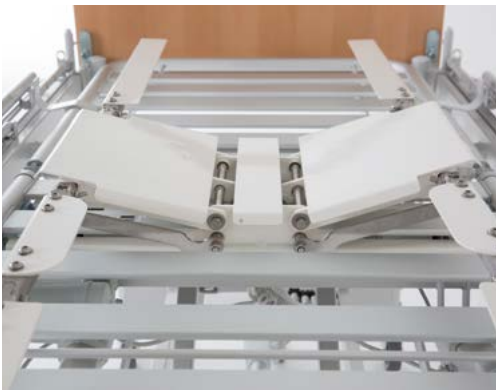

Cradling for comfort

Soft Tilt can be programmed to turn the patient gently from side to side when both wings are elevated.



Intuitive handset

Clear and easy to use handset for manual use.









Cover

Blue cover with elastic straps helps keep the mattress in place. Machine washable up to 80 degrees.

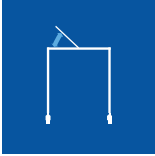Technical data

For more comprehensive pre-sales information about this product, including the product's user manual, please visit www.invacare.co.uk.

	Max angle in manual mode	Max angle in automated mode	Time values in automated mode	Max. user weight	Heaviest part of the product	Cover washable
Soft Tilt	 60 °	 30 °	 0 - 180 minutes	 165 kg SWL 185 kg	 15 kg	 80°

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
NB
The bed must not be used by patients under 12 years of age, or by patients with body size equivalent to an average 12 year old or smaller.
Soft Tilt can only be used in combination with **SB 755**.

Complies with IEC 60601-2-52

2 years warranty

Frame colour

White



Please notice colors may vary slightly from those displayed above.

The **Soft Tilt** is manufactured by GDV Technology on behalf of Invacare.



Visit www.safepatienthandling.co.uk

Invacare Limited
Pencoed Technology Park - Pencoed
Bridgend CF35 5AQ - United Kingdom
Tel: +44 1 656 776 222
Fax: +44 1656 776 220
E-mail: uk@invacare.com
Sales Order E-mail: ordersuk@invacare.com
www.invacare.co.uk

Invacare Ireland Ltd
Unit 5 - Seatown Business Campus
Seatown Road - SWORDS - County Dublin - Ireland
Tel. +353 1 8107084
Fax +353 1 8107085
Email: Ireland@invacare.com
www.invacare.ie

