Invacare® Evacuation Mattress Cover

en	Accessory for mattresses User Manual	1
de	Zubehör für Matratzen Gebrauchsanweisung	3
fi	Patjojen lisävaruste Käyttöohje	5
no	Tilbehør til madrasser Bruksanvisning	8
sv	Tillbehör till madrasserna Bruksanvisning	10

1 General

en

1.1 About This Manual

This document is a supplement to the product's user documentation.

This component itself does not bear a CE and a UKCA mark but is part of a product that complies with the Medical Device Regulation 2017/745, Class I and Part II UK MDR 2002 (as amended) Class I concerning medical devices. It is therefore covered by the product's CE and UKCA marking. See the product's user documentation for more information.

Only use this component if you have read and understood this manual. Seek additional advice from a healthcare professional who is familiar with your medical condition and clarify any questions regarding the correct use and necessary adjustment with the healthcare professional.

Note that there may be sections in this document, which are not relevant to your component, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the component.

Invacare reserves the right to alter component specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the component, for example safety notices and recalls, contact your Invacare representative. See addresses at the end of this document.

In case of a serious incident with the component, you should inform the manufacturer and the competent authority in your country.

Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

Other Symbols

(Not applicable for all manuals)

UK Responsible Person

Indicates if a product is not manufactured in the UK.

Triman

Indicates recycling and sorting rules (only relevant for France).

1.2 Warranty Information

We provide a manufacturer’s warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

2 Product Overview


2.1 Product Description

The Invacare **Evacuation Mattress Cover** is a product designed to allow a quick response in the event of an evacuation of the building. This mattress cover has a built-in quick release webbing which is used to secure the patient in the mattress safely. The head and foot end straps have quick release extendable pull straps to drag both the mattress and the patient safely, eliminating any trip hazard when pulling the patient out of the building.

 The Invacare **Evacuation Mattress Cover** is a single use product. After a emergency evacuation **MUST** be discarded and substituted by a new one.

2.2 Scope of Delivery

The following components are included within the scope of delivery:

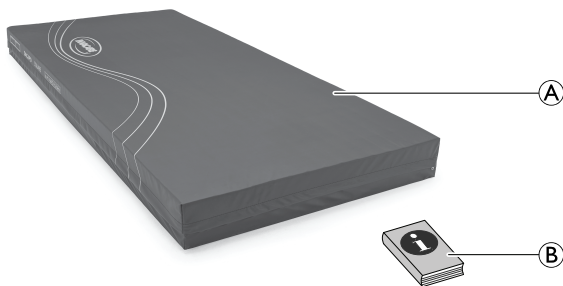A	Evacuation Mattress Cover
B	User Manual

3 Setup

3.1 Preparing the Evacuation Mattress Cover for use

The following steps must be followed to prepare the mattress for evacuation.

WARNING!

Risk of Serious Injury or Damage.


Improper use of this product may cause injury or damage. Before initiating an evacuation procedure:

- Ensure the patient is positioned in the middle of the mattress and that there are no part of the body overhanging the mattress.


1. Release the securing buckle as per visual instruction printed on the top cover.

2. Pull strap away from the securing tabs until the strap is fully extended.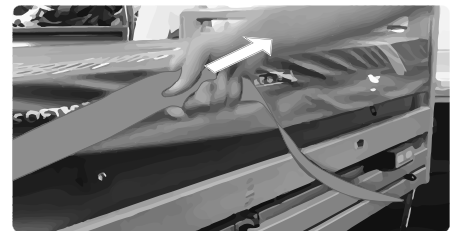


3. Throw the extended strap across the mattress and patient.

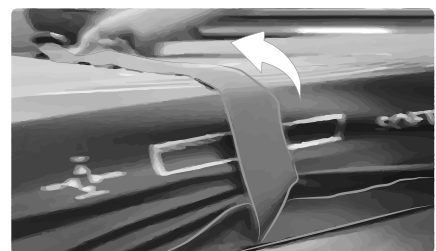
4. On opposite side of the mattress, release the opposite strap.
5. Attach the securing strap male buckle to the female buckle attached to the mattress.


- Pull strap firmly across the patient ensuring the strap is secure across the chest and the patient arms are over the straps and not under.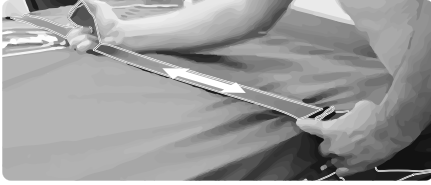

- Repeat steps 2 through to 6 for opposite side on mattress ensuring the straps are secured across the patient thighs.
- When the patient is securely held in the mattress you can release the drag straps located at the head and foot end of the mattress.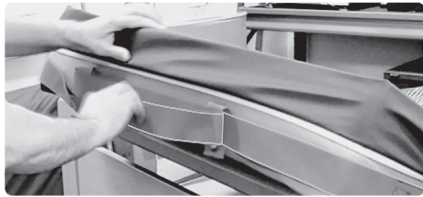

- Pull the drag straps away from the retaining tabs to extend the drag strap.

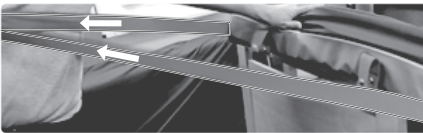4 Usage

4.1 Emergency Evacuation


WARNING!

Risk of Serious Injury or Damage.

Improper use of this product may cause injury or damage. Before initiating an evacuation procedure:


- **ALWAYS** lower the bed to its lowest position.
- Ensure the patient is positioned in the middle of the mattress and that there are no part of the body overhanging the mattress.

Horizontal Evacuation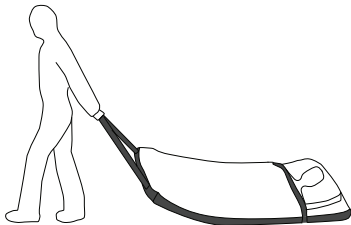

Take a firm hold of the loop drag handle at the foot end, allowing you to evacuate the patient in a dragging motion. It is advisable that a second person uses the drag strap at the head end to navigate more easily around bends.

Vertical Evacuation

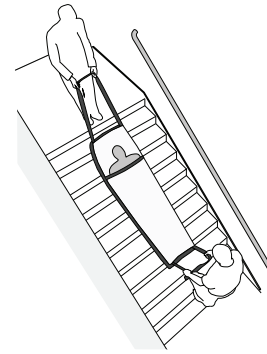If the patient is located on a higher or lower level, you will need to use the stair wells to make your way to the ground floor. Make sure you have enlisted a second person to assist and ensuring you take command of the stairs, the second person can be responsible for any necessary breaking when making your way down the stairs.

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Invacare AB
Invacare Limited
Invacare UK Operations Limited