
# Verso HC

Accent®, Medley® Ergo, Etude Plus, Etude Plus® HC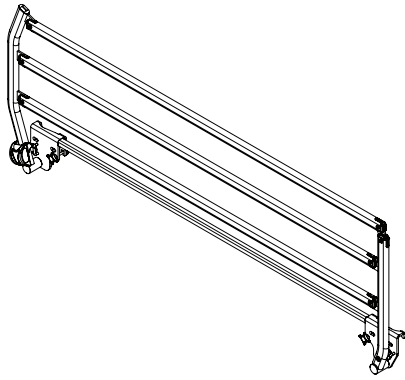

en	Bed side rail User Manual .....	1
fr	Barrière latérale du lit Manuel d'utilisation.....	4
es	Barandilla para cama Manual del usuario .....	7
pt	Guarda lateral da cama Manual de utilização.....	10

## 1 General

### 1.1 About this manual

This manual belongs to an Invacare accessory and it contains important information about handling and assembly. In order to ensure safety when using the product, read the user manual of the related product carefully and follow the safety instructions.

#### Symbols in this manual

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

#### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

#### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

#### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

#### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

### 1.2 Intended use

The side rail is intended:

- to prevent the user from falling out of the bed.
- to be operated by a third person and not by the user himself.
- for home care and long term care facilities only.

#### WARNING!

- Any other or incorrect use could lead to hazardous situations.
- *Invacare®* accepts no liability for any use, change or assembly of the product other than as stated in this User manual.

### 1.3 Service Life

The expected service life of this product is ten years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

### 1.4 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

### 1.5 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

This manual **MUST** be given to the user of the product. **BEFORE** using this product, this manual **MUST** be read and saved for future reference.

**Yes, you can.®**

## Product-specific standards

The product has been tested and conforms to IEC 60601-2-52 (Particular requirements for the basic safety and essential performance of medical beds) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

## 2 Safety

### 2.1 General warnings

#### WARNING!

##### Risk of personal damage

There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.

- Pay attention to your fingers.
- Follow instructions carefully.
- After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

#### WARNING!

##### Risk of entrapment

There's a risk of entrapment or suffocation between mattress support, side rail and bed end.

- Always make sure that the distance between the bed end and the side rail's handle upper edge is less than 6 cm in the head end and more than 32 cm in the foot end of the bed.

#### WARNING!

##### Risk of entrapment or suffocation

There's a risk of entrapment or suffocation between mattress support, side rail and bed end.

- Always ensure correct fitting and tightening of the side rails.

#### WARNING!

##### Risk of falling

If it has been decided that the user needs side rails:

- Never leave the user unattended in the bed with the side rail down.
- Make sure the side rail is in its highest position and properly locked while leaving the user unattended.

#### CAUTION!

##### Risk of Injury

If the side rail is not locked properly it can fall down.

- Pull/push the top bar of the side rail to ensure that the locking system is properly engaged.

#### CAUTION!

##### Risk of Injury

There is a risk of entrapment or squeezing while operating the side rail.

- Pay attention to your fingers and body parts of the patient.
- Never force or drop the side rail while handling it.

## 2.2 Mattresses


#### WARNING!

##### Safety aspects regarding combination of side rails and mattresses:

In order to obtain the highest possible safety level, when using side rails on the bed, it is important to respect the minimum and maximum measures for mattresses.

- For correct mattress measures; see Bed User Manual for the bed in use.

## 3 Components

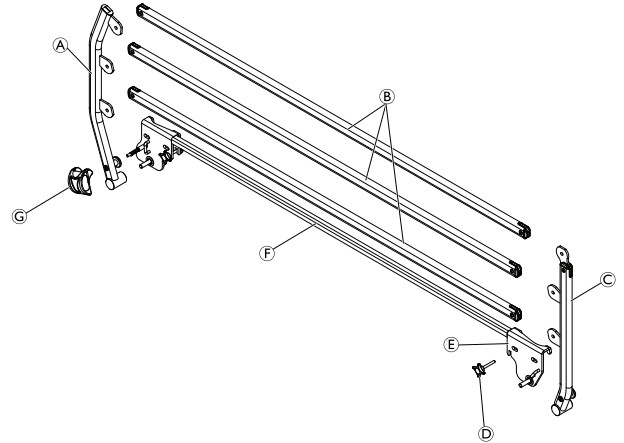- (A) Foot left tube
- (B) Bar
- (C) Head left tube
- (D) Finger screw
- (E) Bracket
- (F) Left base fixation
- (G) Button

## 4 Accessories

#### Important

- Use only original accessories and spare parts. Spare parts lists and extra user manuals can be ordered from *Invacare*.


For repair, please contact your local Invacare® dealer. Service and repair must be carried out by a properly trained technician.

## 5 Setup

### 5.1 Receiving the product

When you receive the product, check the packaging. If the product shows any signs of damage upon delivery, do not use the product.


#### Scope of delivery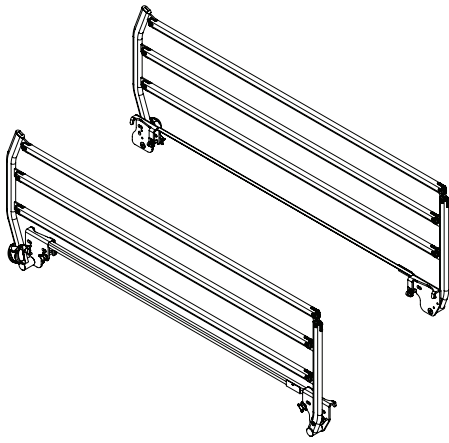

- 1 pair of side rails
- User manual


## 5.2 Installing the side rail

1. Unfold the side rail.
- 2.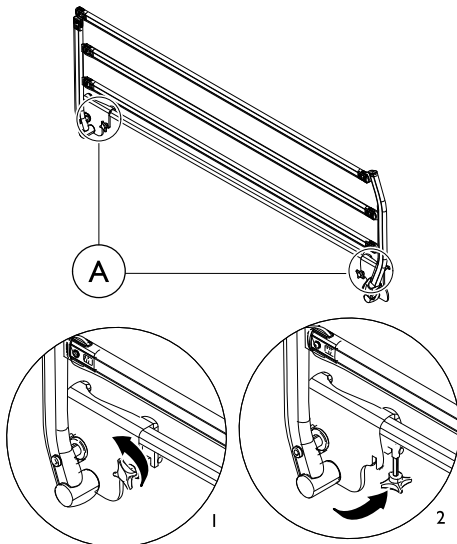


Open the brackets by turning the finger screws **A** anti clockwise (1). Press the fingers screws down and fold them backwards completely out of the notches (2).

3.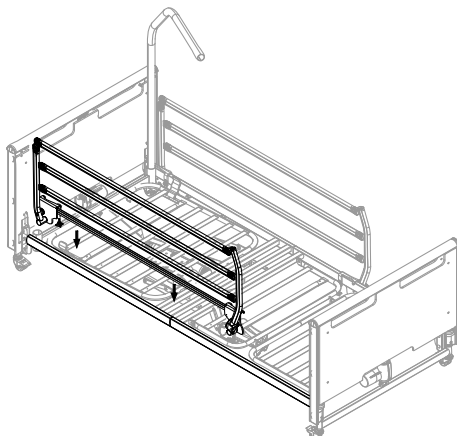

Press the U-formed side rail brackets onto the top frame.

4.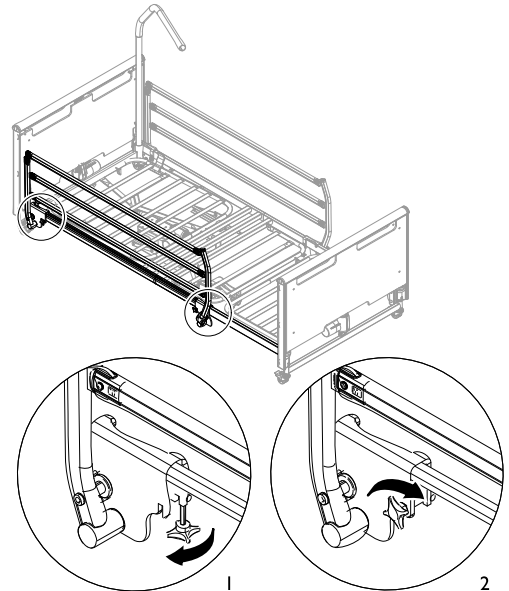


Fold the finger screws forwards (under the top frame) and up into the notches (1). Close the brackets by turning the finger screws clockwise (2).

5. Make sure the finger screws are tightened and the side rail properly attached (not loose).

## 6 Usage

### 6.1 Operating the side rail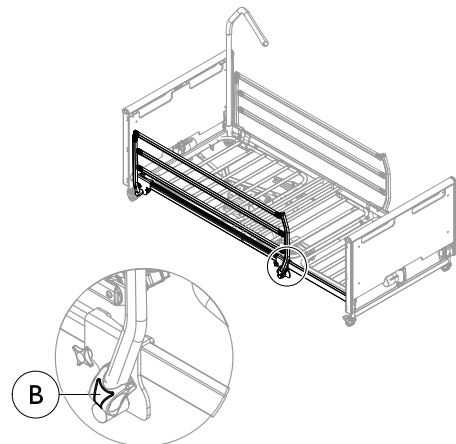

#### Folding down the side rail

1. Hold the top bar of the side rail with one hand and push the release button **B** with the other hand.
2. Fold down the side rail sideways to its lowest position.

#### Folding up the side rail

1. Hold the top bar of the side rail and pull up until it locks with a clicking sound.

## 7 Maintenance

### 7.1 General maintenance information

After 3 months of use, control their fixing, the system of locking and its mechanism. In the event of normal use, the first control of maintenance must be after 2 years then each year.

### 7.2 Cleaning

- Side rails and bed ends should be washed down using a wet sponge, cloth or brush.
- Use ordinary household cleaning agents. Never use acids, alkalines or solvents such as acetone or cellulose thinner.

## 7.3 Waste disposal


Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Waste disposal must comply with the laws and regulations pertaining waste handling in each country.

## 8 Technical Data

### 8.1 Dimensions and weights

All measurements and angles are stated without tolerances, on a non-extended bed.

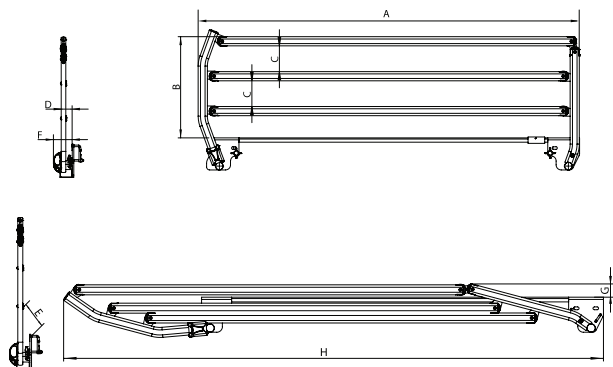Dimensions [mm]		
A	Length	1500
B	Height	397
C	Space between bars	101
D	External dimension out bed	50
E	Space between bed and bar	90
F	External overall dimension out bed	80
G	Height in folding position	45
H	Length in folding position	1945

Weights	
Verso HC side rail — 1 piece	7.4 kg
Mattress dimensions [cm]	
Min. height	14
Max. height	16
Min. width	90
Min. length	190

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Invacare Limited
Invacare Corporation