Invacare® Liv

en Side rail for Invacare® NordBed™ Ultra User Manual	2
da Sengehest til Invacare® NordBed™ Ultra Brugsanvisning	6
no Sengegrind til Invacare® NordBed™ Ultra Bruksanvisning	9
sv Sänggrind för Invacare® NordBed™ Ultra Bruksanvisning	12

This manual MUST be given to the user of the product. BEFORE using this product, this manual MUST be read and saved for future reference.

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Yes, you can.®

1 Introduction

This manual belongs to an Invacare side rail and is an addition to the manuals of the related Invacare bed.

To ensure safety when using this side rail, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.

1.1 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. This document is printed in greyscale. For your information, the safety messages have the following colour coding according to ANSI Z535.6: Danger (Red), Warning (Orange), Caution (Yellow) and Notice (Blue). See the information below for definitions of the signal words.

WARNING!

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION!

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

NOTICE!

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations, and information for efficient, trouble-free use.

2 Installing side rails

WARNING!

Risk of personal damage

There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.

- Pay attention to your fingers.
- Follow instructions carefully.
- After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

2.1 Installing the Liv side rail

WARNING!

Risk of entrapment / suffocation

There's a risk of entrapment or suffocation between side rail and bed end.

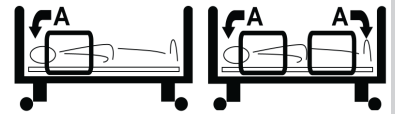
One side rail per side of bed:

- Always make sure that the side rail is mounted close to the head end of the bed and the distance between the bed end and the side rails top bar is 4 cm.

Two side rails per side of bed:

- Always make sure that one side rail is mounted close to the head end and the other close to the foot end of the bed.
- Always make sure that the distance between the bed ends and the side rails top bar is 4 cm.


A=4cm

Mounting

Depending on the individual need of the end-user only one or two side rails can be mounted on each side of the bed.

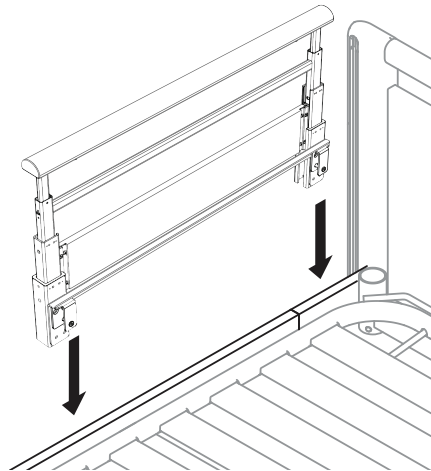
1. Raise the side rail to its high position and remove the two bolts on the brackets.
- 2.

Place both brackets of the side rail on the side tube of the mattress support platform.

3. Position the side rail with a distance of 4 cm between the top bar and the bed end.

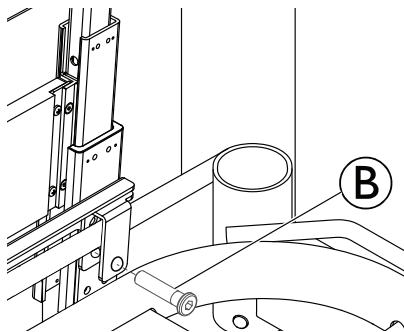4.

Insert the bolt **B** underneath the side tube of the mattress support platform and firmly tighten with a 5 mm Allen key on both brackets of the side rail.

- Repeat the procedure at the head end on the other side of the bed or at the foot end on the same side of the bed, depending on the, desired configuration and availability of additional side rails.

Dismount

- Untighten the bolts and remove on both side rail brackets.
- Lift the side rail of the side tube.

3 Operating the Liv side rail

WARNING!

Risk of falling

The middle position of the side rail does not provide protection against falling out of the bed.

- Only the highest position is intended to reduce the risk for the end-user of unintentionally falling out of the bed.
- The middle positions must only be used to provide support when the end-user is moving in or out of the bed or is being cared.

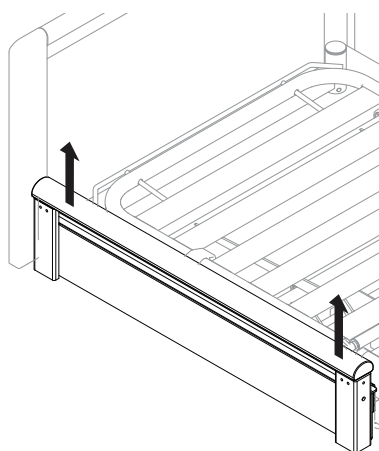
Raising the side rail

NOTICE!


Do not push the release buttons when moving the side rail up.

1.


Pull the top bar upwards with both hands near the left and right end.

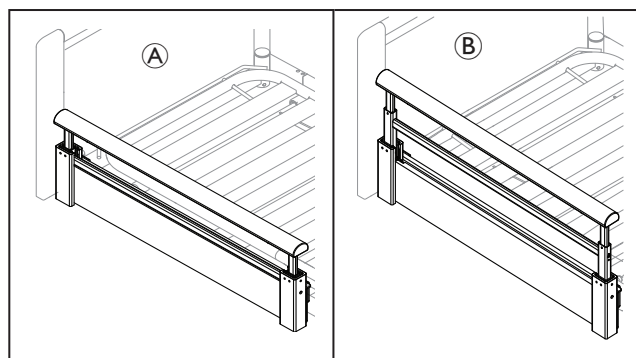2.

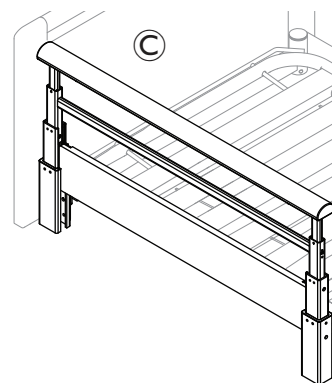To raise the side rail to the middle positions **A** or **B**, stop pulling upwards after the audible click at the desired height.

To raise the side rail to the high position **C**, pull upwards to the upper stop.

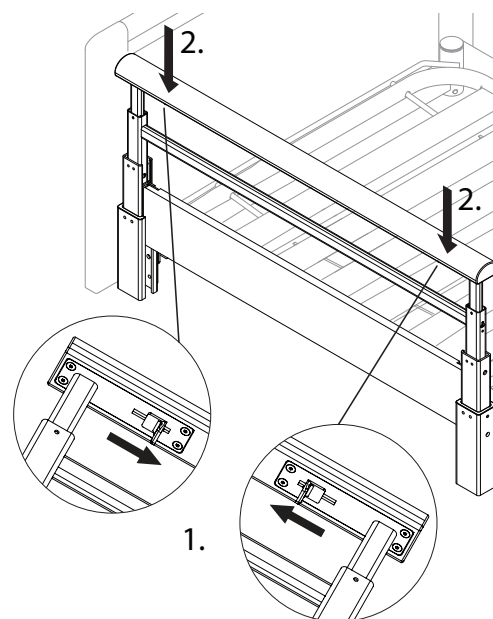
- Push the top bar downwards to ensure it is properly locked.

If the side rail does not lock properly, perform the following steps to restore proper function.

- Pull up the top bar upwards to its stop, without pushing the release buttons.
- Lower the side rail to its lowest position as described below.

Lowering the side rail

- !** **NOTICE!**
Risk of product damage
- Ensure that both release buttons are pushed and held before lowering the side rail.

1. Push up and hold the two release buttons.
2. When lowering, the side rail only stops at the lowest position. To lower the side rail, push the top bar downward.

4 Maintenance


- !** **NOTICE!**
Refer to the Invacare® NordBed™ Ultra User Manual for information on:
- General maintenance information
 - Service intervals
 - Daily inspections
 - Cleaning and disinfection
 - Cleaning intervals
 - Cleaning instructions
 - Disinfection instructions

5 Technical Data

5.1 Materials

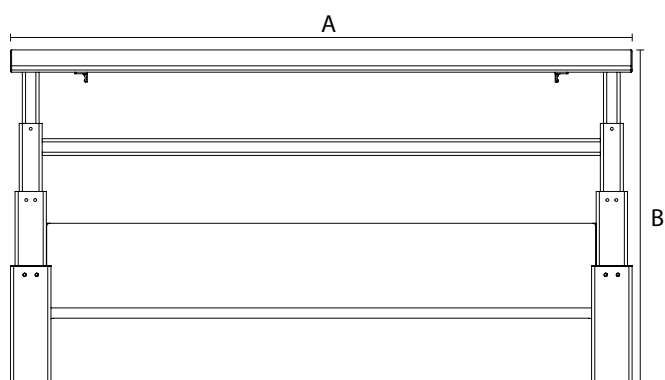Side Rails	Aluminum, Wood, Steel, and Plastic
-------------------	------------------------------------

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Invacare AB
Invacare Rea AB