

Action^{®3} Junior



label 2018
observateur
du design



Your journey, your way



Yes, you can.

Lightweight and **foldable**

The lightweight, foldable wheelchair has been designed with children in mind, encompassing freedom, fun and style with comfort. It's modern design options make it a wheelchair to be proud of and thanks to its integrated width and depth growth ability, the child and the chair can grow together, making it a true companion.

- ▶ Innovative telescopic cross brace
- ▶ Configured for customisation
- ▶ Adjustable seat height and balance




KEY

EXCLUSIVE
FEATURES

1 Innovative telescopic cross brace

The **Action³ Junior** wheelchair can be optimised to the changing shape of the child without having to be replaced, ensuring a perfect and comfortable fit as they grow. This is due to the telescopic cross brace which allows the seat to be adjusted in width and depth. With the seat and backrest upholstery also allowing for quick and easy width adjustability, there is no need for additional parts!

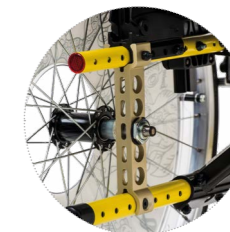

2 Adjustable seat height and balance

This feature of the **Action³ Junior** has been designed to improve the driving performance for the child as it allows the rear wheels to be mounted in multiple Centre of Gravity (CoG) positions and select from three different seat height options: 410, 435 and 560 mm. This ensures an opti-mal set up for propelling the wheelchair.



3

3 Configured for customisation

The wheelchair has been designed to allow the child to personalise their chair to meet not only their needs but also their style. Choose from a selection of soft or vibrant frame colours and then choose one of the 3 modern spoke guard designs. The range of accessories available on the **Action³ Junior** further enhances the customisation of the wheelchair with a range of front wheels, armrests, hangers plus more, available as optional extras.

Features and options ▼



◀ Selection of spoke guard designs

3 modern spoke guard designs to choose from along with 6 frame colours.



◀ Front wheel assortment

3 different types of front wheels available: ① 5" x 1" flashing solid tyre, ② 5" x 1 3/4" solid tyre and ③ 5" x 1" solid tyre.



◀ Armrests

Removable and height adjustable desk armrests ① and junior armrests ② are available as well as skirt ③ and mudguards ④ options.



▲ Hangers

① 90° hanger, ② 80° hanger and ③ elevating legrest, adjustable from 0° to 80°, are available to suit a range of user needs.



▲ Lightweight seat upholstery

Adjustable in width, it allows for easy handling and transportation.

Options ▼




▲ Table tray

In addition to the standard slide-on table tray, the swing-away tray is easy to remove and can be folded away when transferring or not required.



▲ Cushion

The 50 mm bi-density foam cushion provides additional comfort for the user.

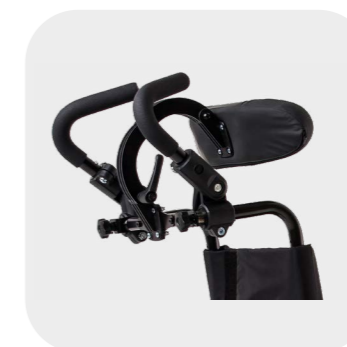

▲ Ergonomic Stroller push handles

The adjustable handles ensure comfort and ease of use for the carer. A width-adjustable back brace can also be added (or used on its own) to make the backrest more rigid and it can be utilised as a headrest support to provide greater comfort to the user.



▲ Abduction legrest position

A 30° abduction position is possible with 80° hangers for those with hip problems.



▲ Transit Version

The transit version has a seat height of 460 mm with 12" rear wheels.



▲ Posture Belt

Available in 2 sizes, it can be adjusted to secure children's positioning. Developed with Bodypoint® with a central adjustment and easy to use buckle.

















▲ Front stabilizer kit






This provides stability for active children who are likely to climb on the wheelchair and for those who have a tendency to be overly enthusiastic when propelling.

Technical data ▼

For more comprehensive pre-sales information about this product, including the product's user manual, please see your local Invacare website.

| |  |  |  |  |  |  |  |
|------------------------|---|---|---|---|---|---|---|
| | Seat width ▼ | Seat depth ▼ | Seat angle ▼ | Seat height ▼ | Backrest angle ▼ | Backrest height ▼ | Front frame angle ▼ |
| Self Propulsion | 205 / 230 / 255 / 280 / 305 / 330 / 355 / 380 mm | 200 - 250 / 250 - 300 / 300 - 350 / 275 - 325 / 325 - 375mm | 1.5° / 5° / 8.5° | 410 / 435 / 460 mm | 0° / 7° / 0° - 30° | 350 - 430 mm | 80° / 90° / 0° - 80° |
| Transit | 305 / 330 / 355 / 380 mm | 250 - 300 / 300 - 350 mm | 1.5° / 5° / 8.5° | 410 / 435 / 460 mm | 0° / 7° / 0° - 30° | 350 - 430 mm | 80° / 90° / 0° - 80° |

| |  |  |  |  |  |  |  |
|------------------------|---|---|---|---|---|---|---|
| | Length of footrests ▼ | Armrest height ▼ | Driving unit width ▼ | Total width, folded ▼ | Total length inc. Footrests ▼ | Total length without footrests ▼ | Total height ▼ |
| Self Propulsion | 125 - 450 mm | 150 - 250 / 180 - 280 mm | Seat width + 190 / + 170 ¹ mm | 315 / 270* mm | 740 - 965 mm | 510 - 740 mm | 870 - 1130 mm |
| Transit | 125 - 450 mm | 150 - 250 / 180 - 280 mm | Seat width + 190 / + 170 ¹ mm | 315 / 270* mm | 900 - 1000 mm | 670 - 770 mm | 870 - 1130 mm |

| |  |  |  |  |  |
|------------------------|--|--|--|--|--|
| | Max. Safe slope ▼ | Radius ▼ | Max. User weight ▼ | Total weight, from ▼ | Mass of the heaviest part, from ▼ |
| Self Propulsion | 12° | 750 mm | 80 kg | 15 / 14.5 ¹ kg | 8.5 / 12 ¹ kg |
| Transit | 12° | 750 mm | 80 kg | 15 / 14.5 ¹ kg | 8.5 / 12 ¹ kg |




1. Transit version

Frame colours ▼



Please note: colours may vary slightly from those displayed above.

Upholstery ▼

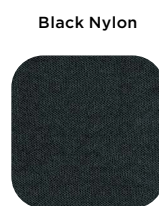

Tel: +41 61 487 70 70
hqeuropa@invacare.com
www.invacare.eu.com

Invacare International GmbH
 Neuhofweg 51
 4147 Aesch BL
 Switzerland



© 2023 Invacare International GmbH. All rights reserved. All information quoted is believed to be correct at time of print. Invacare reserves the right to alter product specifications without prior consultation. EU - 10/23

Yes, you can.®