# Invacare® Actio/2™

P550/2

en	Rollator User Manual .....	2
de	Rollator Gebrauchsanweisung .....	6
es	Rollator Manual del usuario .....	11
fr	Déambulateur Manuel d'utilisation .....	17
it	Rollator Manuale d'uso .....	22
lt	Vaikštinė Naudojimosi instrukcija .....	27
lv	Rolators Instrukciju rokasgrāmata .....	32
nl	Rollator Gebruiksaanwijzing .....	37
pt	Andarilho Manual de utilização .....	42

This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.

**Yes, you can.®**

## 1 General

### 1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

### 1.2 Symbols in this Document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

#### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

#### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

#### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

#### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

### 1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

### 1.4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

### 1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## 1.6 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

## 2 Safety

### 2.1 General Safety Information

#### WARNING!

##### Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

#### WARNING!

##### Risk of Serious Injury

Incorrect use can lead to hazardous situations.

- Do not use on stairs or on escalators.
- Do not stand on the product.
- Be careful when walking on sloping or slippery ground or on uneven surfaces.
- Avoid walking against obstacles, both forwards and backwards.
- Do not let children play with the product.
- Do not use the product to transport a person.
- Make sure that no clothes, such as long skirts, can get stuck in the wheels.

#### CAUTION!

##### Risk of Injury

Unbalanced or heavy load may lead to overturning.

- Do not hang any loads, such as bags, to the frame, the push handles or hand knobs.
- Only carry loads with the designated equipment, like basket or tray. Do not exceed the maximum load for the equipment.
- The total load, including the weight of the user and any additional load, must not exceed the maximum user weight.
- Refer to chapter 7 *Technical Data, page 6* for maximum user weight and load information.

#### CAUTION!

##### Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalog or website for available options or contact your Invacare distributor. See addresses at the end of this document.

### CAUTION!

#### Risk of First Degree Burns

Parts of the product may heat up, when exposed to high temperatures (above 41° C).

- Be careful if the product is exposed to high temperatures, e.g. direct sunlight.


### IMPORTANT!

There is a risk of damage due to wear, e.g. corrosion.

- Do not use the product under water, e.g. in pools.
- Do not use the product in hot areas like saunas.
- Do not store the product in wet or humid environments.

## 2.2 Labels and Symbols on the Product

### Identification label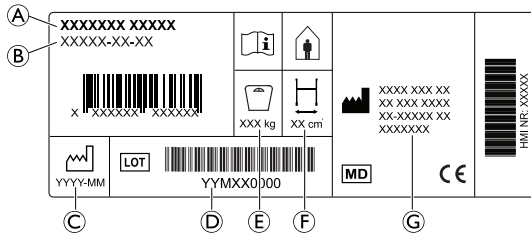

- Ⓐ Product name
- Ⓑ Part number
- Ⓒ Date of manufacture
- Ⓓ LOT number
- Ⓔ Maximum user weight
- Ⓕ Maximum width
- Ⓖ Manufacturer address

### Symbols

	European Conformity		Medical Device
	Manufacturer		Read user manual
	Date of manufacture		LOT number
	Maximum width		Maximum user weight
	Indoor use		

## 3 Product Overview

### 3.1 Intended Use

The walking aid is intended to support a person during walking.

The walking aid is for indoor use only.

The weight of the user must not exceed the maximum user weight as stated on the product label and in chapter 7 *Technical Data*, page 6.

### 3.2 Indications for Use

#### Indications


Persons with impaired muscle strength and/or balance, who need additional support whilst walking. The remaining

walking and cognitive ability must be sufficient for safe use of a walking aid.

#### Contraindications

Persons with severe equilibrium disorders or functional perceptual disorders.

### 3.3 Scope of Delivery

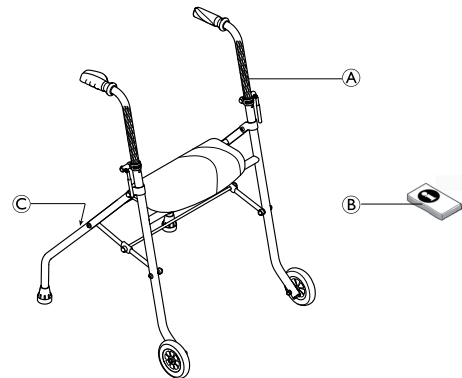- Ⓐ Rollator, partly assembled
- Ⓑ User manual
- Ⓒ Location of the serial label

## 4 Usage

### 4.1 Before Use

#### CAUTION!

##### Risk of Injury or Damage to Property

- Before use, verify the general functionality of the product.
- Before use, check the product for any defect.
- Do not use a defective product.

Invacare presume a healthcare professional has made an individual assessment of the user to determine his risk level or therapy requirements. Providers and carers have assessed that the product gives the required support and that its adjustments are appropriate for the user. Carer and user have been appropriately trained by a healthcare professional on how to use the product correctly and safely.

### 4.2 Operating position


#### WARNING!

##### Risk of falling


- Make sure that the folding mechanism is in a locked position before using the rollator.

1. Remove the plastic caps in the side tubes, if any.
- 2.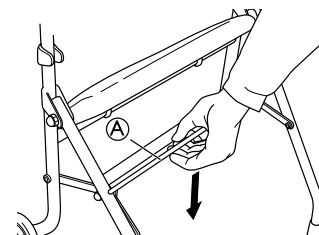


Open the rollator and press the folding mechanism downwards for operating position.

3.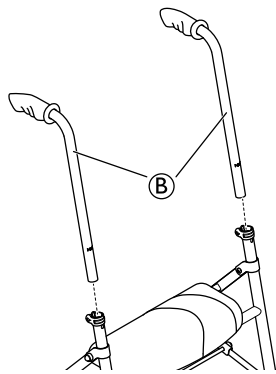


Assemble the push handles B.

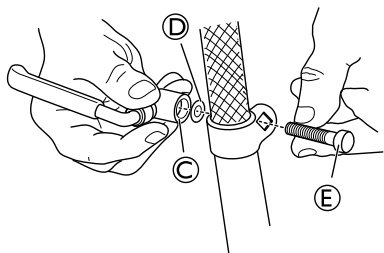4. Assemble the lever C, the washer D and the bolt E with the locking lever on the outer side of the tube.


 Ensure that the bolt fits correctly into the square shaped hole on the attachment.

5.

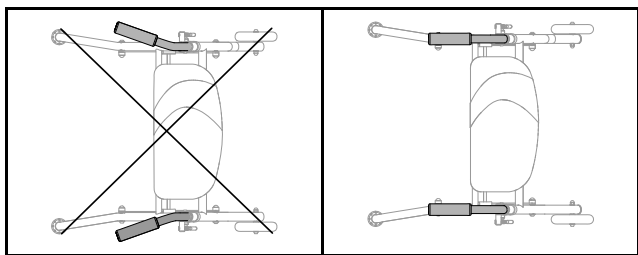 **WARNING!**  
**Risk of falling**  
– The handlebars must not be angled outwards as this reduces the lateral stability of the rollator.

6.

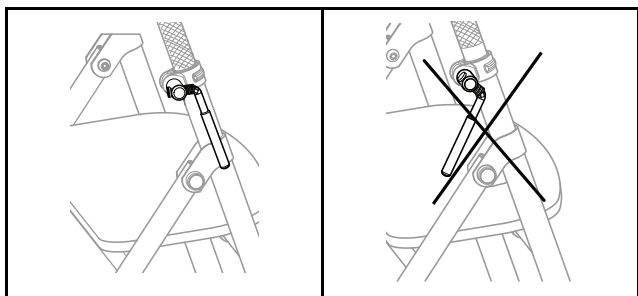Tighten the locking lever.


 The locking levers must be pointing downwards during use.

7. Repeat the procedure on the opposite side.

### 4.3 Unfolding and folding the rollator


#### Unfolding the rollator

 **WARNING!**  
**Risk of falling**  
– Make sure that the folding mechanism is in a locked position before using the rollator.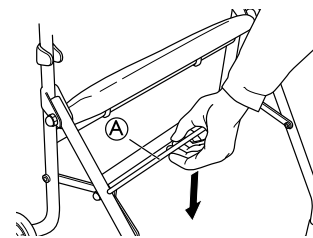


1. Unfold the rollator.
2. Push the folding mechanism A downwards.

#### Folding the rollator

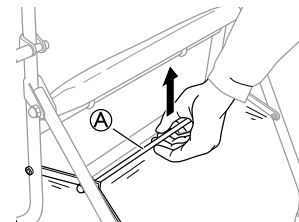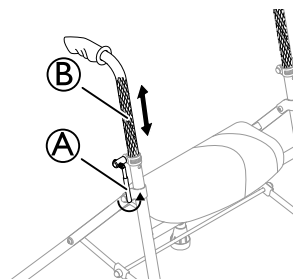
1. Pull the folding mechanism A upwards.
2. Fold the rollator.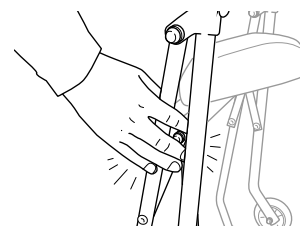


**WARNING!**  
**Risk of pinching**  
– Be aware of the pinching risk when folding the rollator.

### 4.4 Adjusting the handle height/angle

 The handles should usually be positioned at wrist height. Therapist may suggest different heights depending on specific needs of the client.


1. Stand behind the rollator with flat shoulders.
2. Loosen the lever A.
3. Adjust the handle B to suitable height.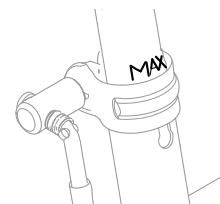

**WARNING!**  
**User may fall**  
– Make sure that the push handles are not adjusted more than maximum. See the marking on the push handle tube.

4. Re-tighten the lever.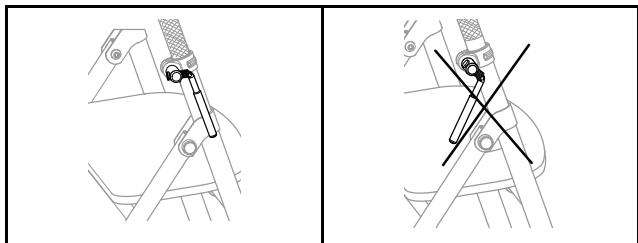


**i** The locking levers must be pointing downwards during use.

5. Repeat the procedure on the other side.

## 4.5 Using the seat


The seat cushion can be used as a resting seat.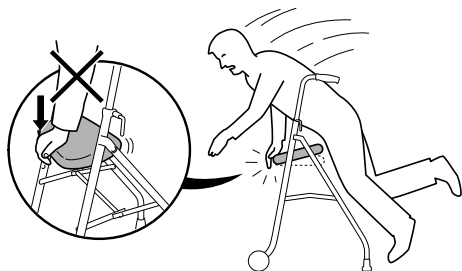

### **WARNING!** Risk of falling

- Do not lean on or sit on the front edge of the seat as the seat may open.

## 4.6 Mounting the basket

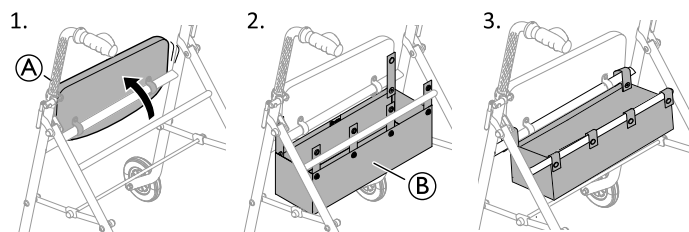1. Fold up the seat cushion **A**.
2. Place the basket **B** between the cross tubes.
3. Fold the attachment straps around the cross tubes and close the buttons.
4. Fold the seat back down again.

**i** See section “Dimensions and weight” for information about max load capacity.

## 5 Maintenance

### 5.1 General Maintenance Information

### **WARNING!** Risk of Injury

- Maintenance should be done at least once a year.
- Do not use a defective product.
- Do not attempt repairs yourself. For repairs contact your provider.

### 5.2 Inspection Checklist

To ensure good performance and safety, inspect your product regularly.

- Ensure that all screws and adjustments are properly tightened.

- Ensure that hand grips are tight. If hand grips are loose do not use the walking aid. Instructions on how to change hand grips can be obtained from your provider or Invacare distributor.
- Verify brake operation. If brake adjustment is required, see chapter 4 *Usage*, page 3.
- Inspect the frame and loaded parts for cracks.
- Inspect the wheels’ tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

## 5.3 Cleaning and Disinfection

### General Safety Information

### **CAUTION!**

#### Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

### **IMPORTANT!**

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

### Cleaning Intervals

### **IMPORTANT!**

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product

- regularly while it is in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

### Cleaning Instructions

The product can be washed in a washing machine or by hand.

When using a high-pressure cleaner or an ordinary jet of water, do not aim the jet towards bearings, draining holes or labels.

The maximum cleaning temperature is 60 °C. Do not use a steam cleaner.

1. Clean the walking aid with a damp cloth and mild detergent.
2. Clean the wheels with a brush (no wire brush).
3. After washing, wipe the product dry.

### Disinfection

- Wipe down all generally accessible surfaces with disinfectant.

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

## 6 After Use

### 6.1 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

### 6.2 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection


For detailed information, see *5 Maintenance, page 5* and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

## 7 Technical Data

### 7.1 Dimensions and weight

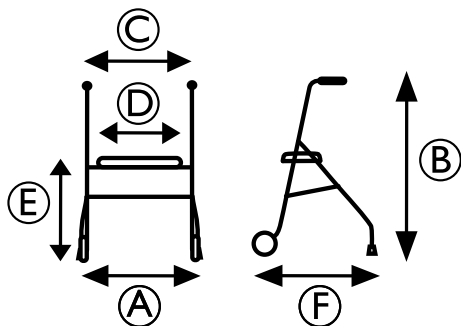	<b>P550/2 ACTIO 2</b>
Total width <b>A</b>	530 mm
Total height <b>B</b>	740 — 1000 mm
Total width between push handles <b>C</b>	410 mm
Total seat width <b>D</b>	380 mm
Total seat height <b>E</b>	550 mm
Total length <b>F</b>	630 mm
Maximum user weight	130 kg *
Total weight without accessories	5 kg
Maximum load basket	2,5 kg
Diameter front wheels	110 mm

\* The maximum user weight is the weight limitation of the product and includes the contents of the basket and the weight of the user.

## 7.2 Material

Frame	Polyester powder coated steel
Handle bars	Chrome plated steel
Handgrips	PVC
Wheels	Polypropylene with treads of TPE
Screws	Steel, rust protected

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All rights reserved. Republication,
Trademarks are identified by ™ and
®. All trademarks are owned by or
Invacare AG
Invacare Limited
© 2019 Invacare Corporation.