Aquatec® Ocean / Ocean VIP/ Ocean Dual-VIP / Ocean E-VIP

- EN** Chest and pelvic belt
User Manual
- DE** Brust- und Beckengurt
Gebrauchsanweisung
- FR** Ceinture thoracique et abdominale
Mode d'emploi
- IT** Cintura pettorale ed addominale
Istruzioni per l'uso
- ES** Cinturón pectoral y pélvico
Manual de instrucciones
- NL** Borst- en bekkengordel
Gebruiksaanwijzing
- DA** Bryst- og bækkenbæltet
Brugsanvisning
- FI** Rinta- ja lantiovyön
Käyttöohje
- NO** Bryst- og hoftebeltet
Brukerveiledning
- SV** Bröst-/bäckenbältet
Bruksanvisning
- PT** Cinto torácico e pélvico
Manual de instruções

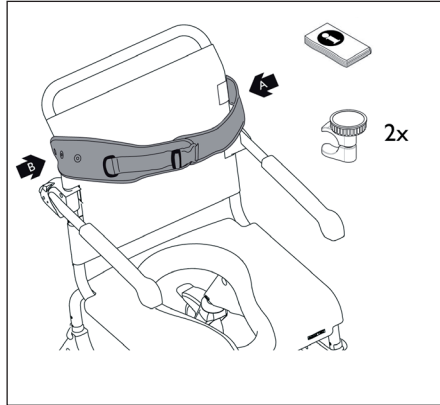
Fig. 1

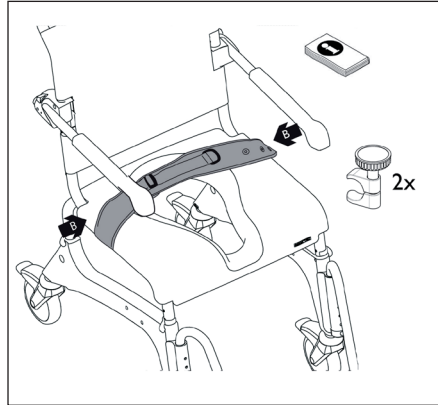
Fig. 7

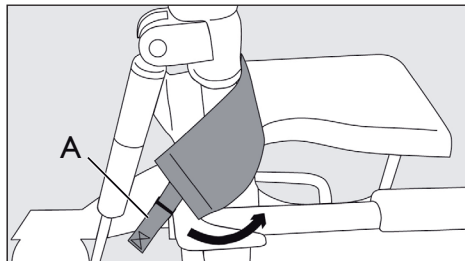
Fig. 2

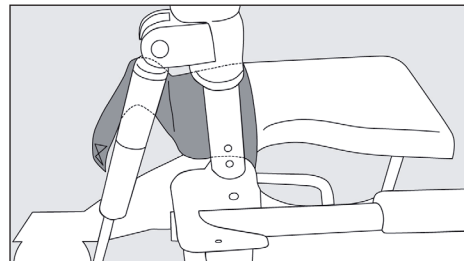
Fig. 3

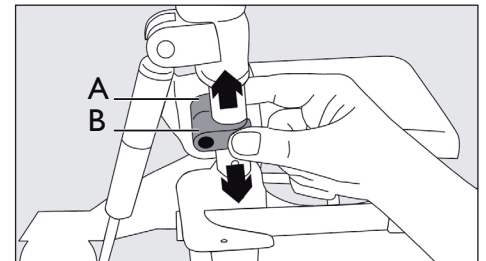
Fig. 4

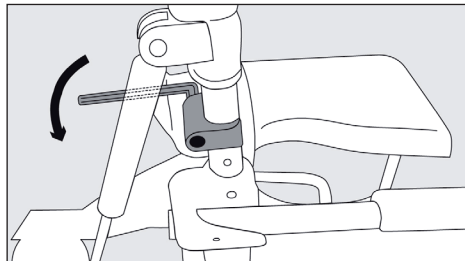
Fig. 5

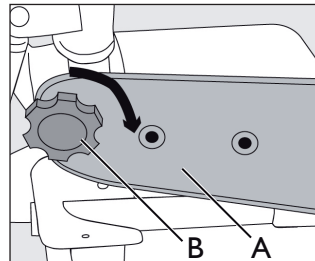
Fig. 6.1

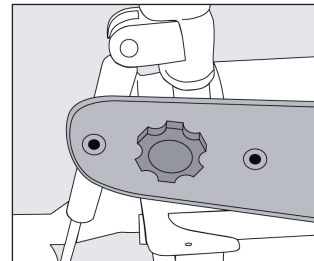
Fig. 6.2

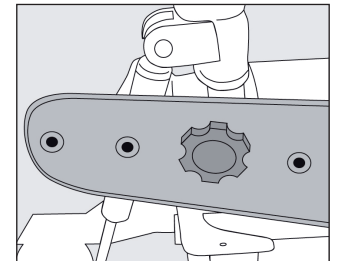
Fig. 6.3

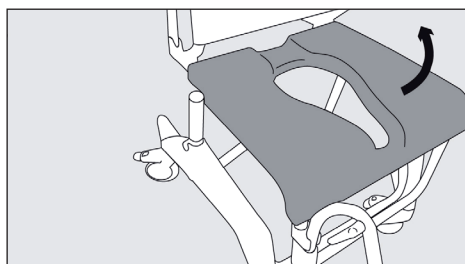
Fig. 8

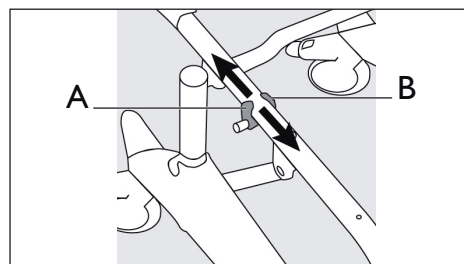
Fig. 9

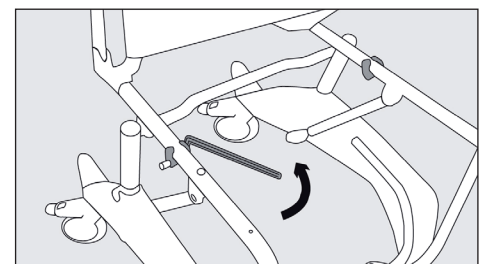
Fig. 10

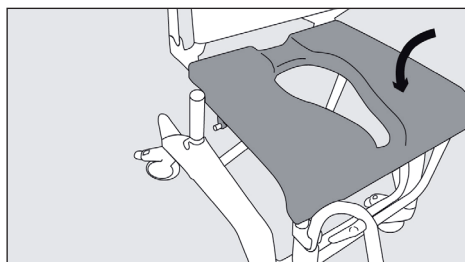
Fig. 11

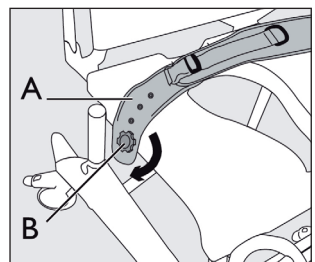
Fig. 12.1

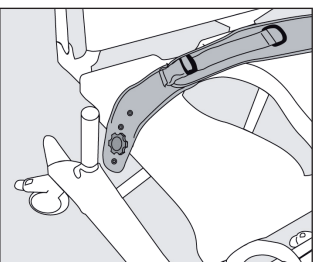
Fig. 12.2

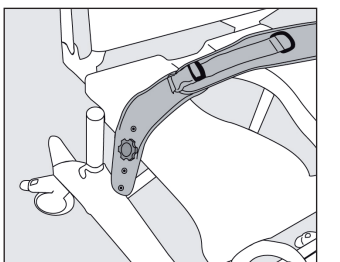
Fig. 12.3

This manual must be given to the user of the product.
Before using the product, read this manual and save it
for future reference!

AQUATEC®

Yes, you can.

Intended use

The chest and pelvic belt is intended solely for use with the Aquatec® Ocean Range for secure patient positioning. Any other use is prohibited.

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

Service Life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

WARNING Risk of injury!

- ▶ Do not use if defective.
- ▶ Check product regularly for damage and to ensure that it is securely assembled.
- ▶ Adjust the position and length of the chest/pelvic belt on the user.
- ▶ Fasten belt after use. The chair may brake abruptly and inadvertently due to the belt hanging loosely downwards. The fastening mechanism of the belt hanging downwards may be destroyed if driven over.
- ▶ Tighten screws and regularly check that they are secure.
- ▶ Be careful during assembly.
- ▶ The material is not flame-retardant.

Scope of delivery

- ▶ Fig. 1 bzw. Fig. 7.

Fit and adjust the position of the chest and pelvic belt

- ▶ Fig. 2 - 6 bzw. Fig. 8 - 12.

Adjust the length of the chest and pelvic belt

- ▶ Fig. 6.1/2/3 bzw. Fig. 12.1/2/3.

Cleaning

Note the cleaning instructions on the product label.

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AQUATEC Operations GmbH