
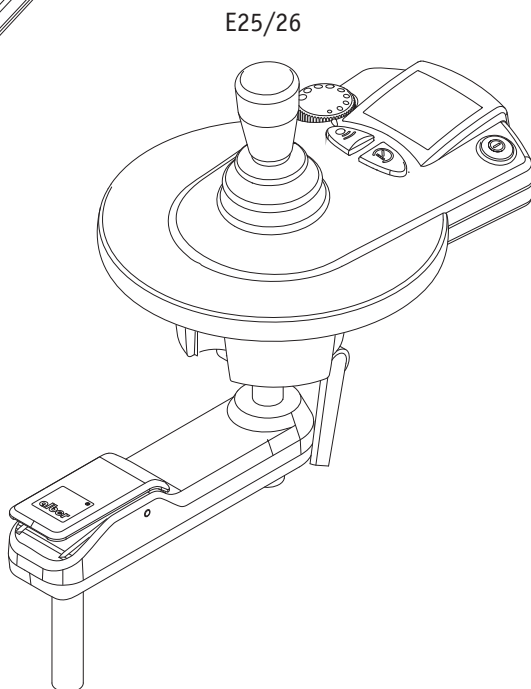
Gebrauchsanweisung Schwenkarm e-fix
Instructions for use swivel arm e-fix
Instructions d'utilisation du bras pivotant e-fix

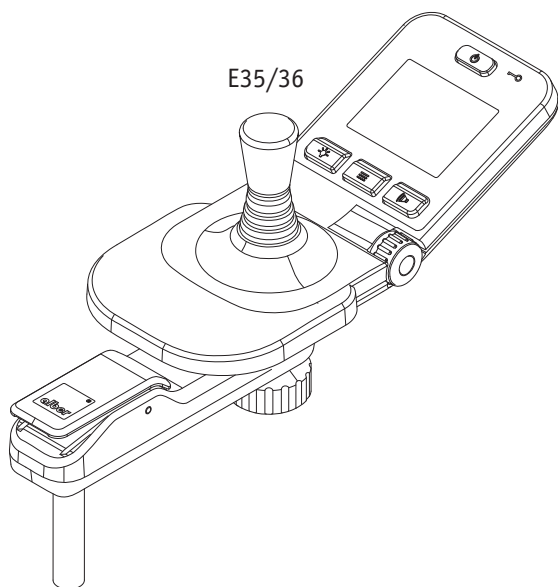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

Example of use for swivel arms e-fix E25 and E35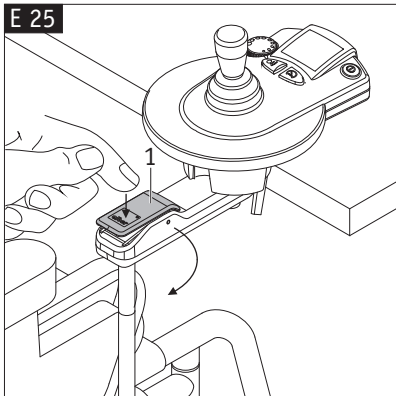

Approaching a table

- When approaching the edge of a table with your wheelchair, remain far enough from the table so that it is still possible to move the swivel arm.
- Press the control button [1] and swing the swivel arm to the desired lateral position. At an angle of approx. 45 degrees you will hear a soft “clicking” sound. If the swivel arm is swung back to its starting position from this position, it then locks back into this position automatically.
- Now move closer to the table at low speed and exercise caution in doing so.
- Switch off the control unit if it is not being used!

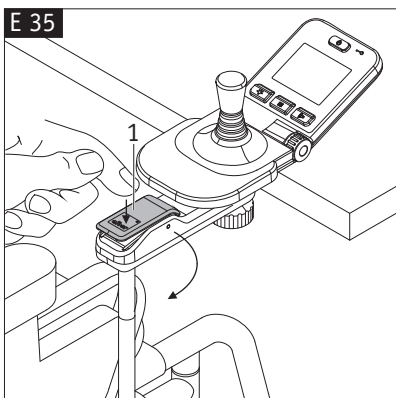
Moving away from a table

- Switch on the control unit if necessary.
- Move back slightly from the table and do so slowly.
- Swing the swivel arm back to its starting position. It will lock into this position automatically.
- You can now continue to move as usual


Always switch off the control unit if it is not being used! This prevents your wheelchair from rolling away unintentionally, e.g. due to accidental contact with the joystick.

See the operating manual for the e-fix E35 regarding attachment and removal of the control unit E25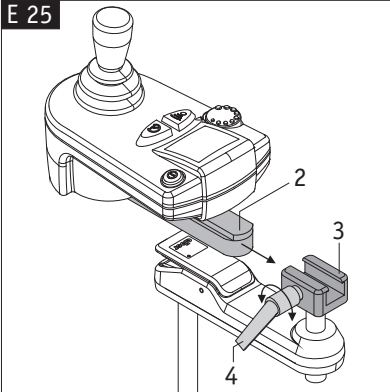

Attaching the control unit E25

- Push the control unit with the guide rail [2] into the mount [3] **without** the connection cable [5] coming from the interface.
- Slide the control unit into the position you want and fix it in place by turning the clamping lever [4].
- Now connect the connection cable [5] to the control unit.

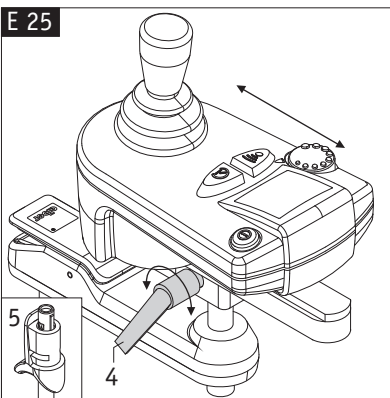Removing the control unit E25

- Switch off the control unit.
- Remove the connection cable [5] from the control unit.
- Loosen the locking lever [4] and pull the control unit out of the mount [3].

Always switch off the control unit prior to attachment or removal! This prevents your wheelchair from rolling away unintentionally, e.g. due to accidental contact with the joystick.

Always ensure that the control unit does not fall onto the ground! If the control unit is switched on and it falls to the ground, this could trigger an unintentional travel impulse.

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Alber GmbH