# Matrix® SHAPE INSERT

FOR USE WITH THE **MATRIX MAC-HUG BACK**

|    |                                             |    |
|----|---------------------------------------------|----|
| en | Backrest<br>User Manual . . . . .           | 5  |
| de | Rückenlehne<br>Gebrauchsanweisung . . . . . | 11 |
| fr | Dossier<br>Manuel d'utilisation . . . . .   | 17 |
| it | Schienale<br>Manuale d'uso . . . . .        | 23 |
| es | Respaldo<br>Manual del usuario . . . . .    | 29 |
| pt | Encosto<br>Manual de utilização . . . . .   | 35 |
| nl | Rugleuning<br>Gebruiksaanwijzing . . . . .  | 41 |
| da | Ryglæn<br>Brugsanvisning . . . . .          | 47 |
| no | Rygglene<br>Bruksanvisning . . . . .        | 54 |
| sv | Ryggstöd<br>Bruksanvisning . . . . .        | 60 |
| fi | Selkänöja<br>Käyttööpas . . . . .           | 66 |

**en** This manual may not be reproduced or reprinted either partly or completely without previous written consent from Motion Concepts or its statutory representatives.

This user manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make any necessary as they become necessary. Any changes made to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. If you have any questions or concerns regarding the information provided in this manual, please contact our Customer Service Department for assistance.

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**DEALER:** This manual **MUST** be given to the user of the product.

**USER:** Before using this product, read this manual and save for future reference.

## 1.0 GENERAL WARNINGS/INFORMATION

### 1.1 OVERVIEW:

#### SAFETY SYMBOLS:

Proper set-up and safe use of your Matrx® Shape Insert depends on your own good judgement and common sense, as well as that of your provider, caregiver and/or health professional. The following symbols are used throughout this user manual to identify warnings, cautions and important notes. It is very important that you read and understand them completely.

 **CAUTION!** Failure to heed the cautions in this manual may result in minor injury or damage to your Matrx Seating product, or both.

 **WARNING!** Failure to heed the warnings in this manual may result in death, serious injury or damage.

 **IMPORTANT!** Important information to remember during installation, set-up and/or adjustment of your Matrx Seating product.

 **WARNING! Risk of serious injury or damage**

Improper use of this product may cause serious injury or damage. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a health care professional or dealer before attempting to use this equipment.

 **IMPORTANT INFORMATION:** The best way to avoid problems related to pressure injuries is to understand their causes and your role in a skin management program. Your therapist and physician should be consulted if you have questions regarding individual limitations and needs.

All back supports should be selected carefully. Working with your therapist and physician is the best way to assure that a back/cushion choice matches your individual needs. As the needs of the individual become more complex, the evaluation becomes more important. Regular evaluations should be performed to address any ongoing physical changes.

 **IMPORTANT!** Please refer to the Wheelchair Owners/Operators Manual for additional safety information, warnings and cautions related to the operation of your wheelchair.

### 1.2 INTENDED USE

The Matrx® SHAPE is designed to be used primarily with the Matrx® MAC HUG Back, to accommodate moderate postural asymmetries. Normal use is defined by Motion Concepts as when the product is placed between the padding and rigid back shell.

## 2.0 SAFETY INFORMATION & WARNINGS

### **WARNING! Risk of Serious Injury or Damage**

- The instructions in this manual should be performed by a qualified therapist/technician
- For instructions on the installation/set-up of the Matrix® MAC Back, please refer to the **Matrix® MAC Back User Manual** (provided separately).

### **WARNING! Risk of Serious Injury or Damage**

The Matrix® Shape Insert is NOT a suitable replacement for a back support cushion. NEVER use the Shape Insert on its own without a Back Support Cushion in place.

### **WARNING! Risk of Death, Serious Injury, or Damage**

Lighted cigarettes dropped onto an upholstered seating system can cause a fire resulting in death, serious injury or damage. Wheelchair occupants are at particular risk of death or serious injury from these fires and resulting fumes because they may not have the ability to move away from the wheelchair.

- DO NOT smoke while using your wheelchair

### **WARNING! Risk of Serious Injury**

When establishing the final mounting position for your Matrix® MAC Back, ensure the stability of the wheelchair is not compromised. Adjusting the forward and aft position and/or angle of the backrest will alter the user's center of gravity within the wheelchair. It may be necessary to reposition the front casters, rear wheels, back angle, tilt-in-space, recline position and/or seat depth before use.

### **WARNING Risk of Serious Injury**

Skin condition should be checked very frequently after the installation of any new seating system. Working with your therapist, physician and service provider is the best way to assure that a seating choice meets your individual needs.

### **WARNING! Risk of Injury or Damage**

Matrix® products are specifically designed and manufactured for use in conjunction with Matrix® accessories. Accessories designed by other manufacturers have not been tested by Motion Concepts and are not recommended.

---

## 3.0 WARRANTY INFORMATION

### **WARRANTY**






We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

## 4.0 MATRX SHAPE INSERT


### 4.1 ASSEMBLY:

The Matrix® SHAPE INSERT is comprised of a Fabric Interface Pad (C), a set of 25 polyurethane Foam Blocks (A), and a corresponding set of stretchable Fabric Loops (B). The Velcro™ compatible fabric interface pad may be cut to any desired shape/size without fraying (fig 1). The individual foam blocks are designed to fit inside the fabric loops (fig 2). The assembled foam blocks can be arranged and/or stacked onto the fabric interface pad in any desired configuration via hook & loop fasteners (fig. 3/4).

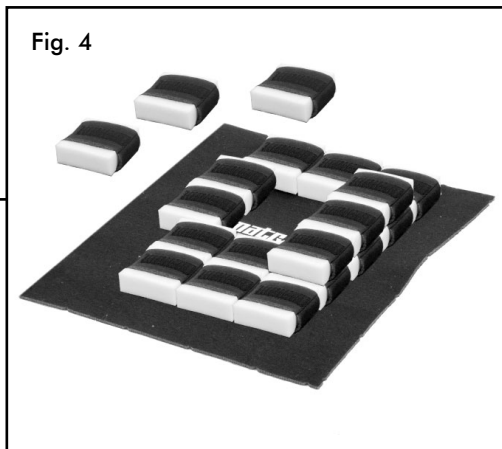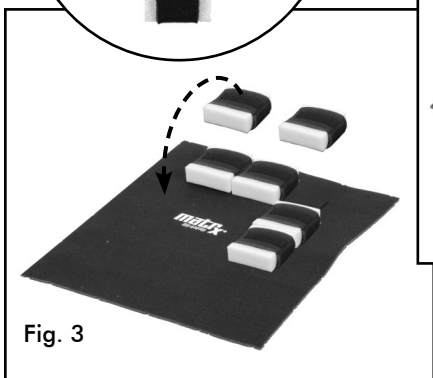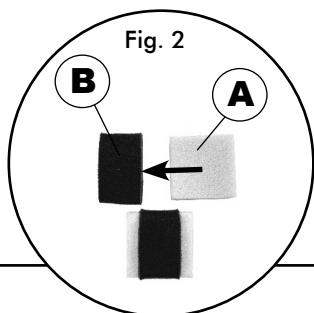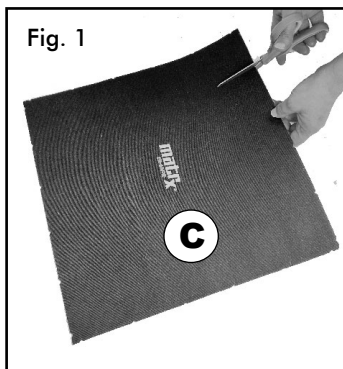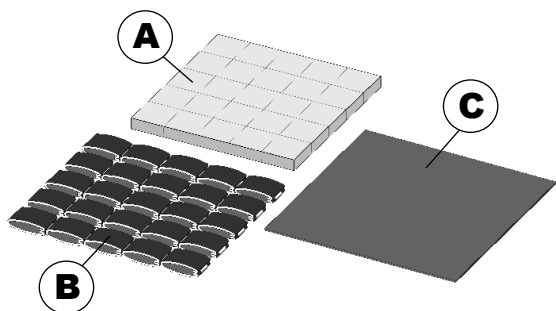**i IMPORTANT!** The Matrix® Shape Insert must be secured to the inside of the backshell via the existing the hook and loop strips. The Shape Insert must be installed BEHIND the Matrix® MAC-HUG Cushion (or other suitable back cushion) -see section 4.2; **NEVER** install in front of the existing cushion or use on its own.

## 4.2 INSTALLATION (ONTO THE MATRIX® MAC-HUG BACK)
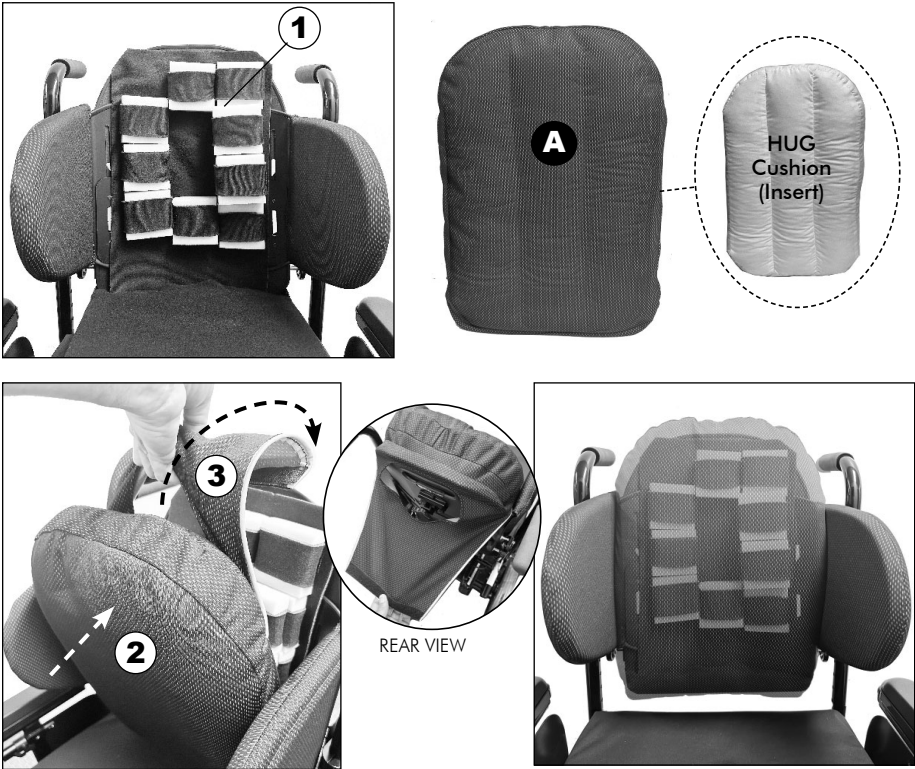
## 5.0 CARE AND MAINTENANCE

### 5.1 INSPECTION AND MAINTENANCE

Frequent visual inspection of the Matrix Shape Insert, as well as all associated Matrix Back components, should be performed to identify deformation, wear and tear or compression. Replace damaged/worn components if necessary.

#### **⚠ WARNING! Risk of injury or damage**

- Do not continue to use this product if problems or damage are discovered;
- Corrective maintenance can be performed by or arranged thru your local service provider

**Weekly:** Inspect **ALL** Matrix Back Support fasteners weekly to ensure that mechanical connections and attaching hardware are tightened securely.

**Monthly:** Perform a visual inspection of all components including hardware, brackets, upholstery materials, foam, and plastics for deformation, corrosion, breakage, wear or compression. Replace damaged/worn components if necessary.

## 5.2 CLEANING/LAUNDERING INSTRUCTIONS:

**i PLEASE NOTE:** Foam blocks develop a yellow tinge over time if exposed to air. The color of the foam is NOT an indicator of the cleanliness of the product.

### **⚠ WARNING! Risk of injury or damage**

- Inspect the Matrix® Shape Insert regularly and launder any fabric components which have become contaminated or soiled.
- DO NOT continue to use foam blocks if they are soiled. Discard any soiled foam blocks in accordance with local government regulation regarding medical waste.

### **⚠ CAUTION! Risk of damage**

- If the Shape Insert becomes exposed to liquids, remove it from the backrest, separate the foam blocks & fabric interface pad, and air dry all components.
- Adequate drying is a crucial factor in the care and maintenance of the Matrix® Shape Insert and the back support. All components should be dried completely prior to reinstallation.

### **⚠ CAUTION! Risk of damage**

- DO NOT use cleaning and disinfecting agents with abrasive, staining, or polymer-damaging properties such as phenols, alcohols, or bleaches. Chlorine solutions used even in low concentrations on a regular basis can diminish the life of the cover/fabric
- Keep components clear of open heat sources during cleaning.
- NEVER use disinfectant agents or wipes to clean the Foam blocks.
- DO NOT use fabric softeners.

### 5.2.1 General Cleaning:

The Matrix® Shape Insert can be wiped with a damp absorbent towel, cloth, or paper towel. Remove all gross contamination from surface (i.e. food particles, urine, feces, blood). Spot-clean with mild soap and water. Ensure ALL components are completely dried prior to reinstallation.

### 5.2.2 Laundering Instructions

The fabric interface pad and fabric loops may be laundered, if necessary. **PRIOR** to laundering, please read all **Warning Information** in section 5.2


#### **LAUNDERING THE INTERFACE PAD AND FABRIC LOOPS:**

1. Remove all gross contamination from the surface using an absorbent material (cloth or paper towel).
2. Remove the Matrix® Shape Insert from the back support and inspect thoroughly.
3. Remove ALL foam blocks **PRIOR** to laundering the fabric interface pad and/or fabric loops (*Discard any soiled foam blocks in accordance with local government regulations regarding medical waste*) **Note:** Replacement Foam Blocks are available for purchase, if required- see 5.2.3 below

4. It is recommended to place the fabric loops and interface pad in a Laundry Bag.
5. Wash fabric components using a mild/neutral detergent on a gentle cycle (*Fabric softener may damage the properties of the foam core layer and is NOT recommended*)
6. Due to foam core layer in the fabric it is recommended to double rinse components after laundering.
7. Hang fabric components to dry. (Air Drying is the preferred drying method, DO NOT tumble dry). Ensure ALL components are COMPLETELY dried prior to reinstallation.

### 5.2.3 Replacement Foam Blocks for the Matrix Shape Insert

Replacement Foam Block sheets may be purchased separately, if required. (1 sheet = 25 pre-cut blocks (p.n. **SHAPE-RF**)). Please contact your local service provider for assistance.

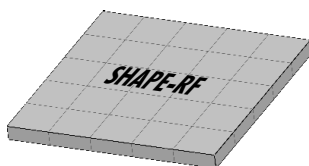## 6.0 AFTER USE

### 6.1 STORAGE

#### **CAUTION! Risk of Damage**

- Store product inside a protective cover whenever possible, in a clean, dry environment away from any sharp edges.
- DO NOT store other items on top of this product,
- DO NOT store this product next to radiators or other heating devices.
- Protect the product from direct sunlight.

### 6.2 REUSE/RECYCLING

This product is suitable for re-use. The Matrix® Shape Insert (fabric and foam), must be inspected by a qualified dealer or healthcare professional before re-use.

#### **WARNING! Risk of Serious Injury**

- Always have an authorized dealer inspect the product for damage or wear PRIOR to transferring the product to a different user.
- If any damage is found, DO NOT RE-USE this product. Failure to heed these warnings may result in serious injury.
- PRIOR to reuse, clean/disinfect the product thoroughly in accordance with local government regulations.

### 6.3 DISPOSAL

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

Ensure that the product is cleaned prior to disposal to avoid any risk of contamination.

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Invacare Limited
Advena Ltd.