
Invacare® Freya / Embla

Invacare® Freya, Invacare® Embla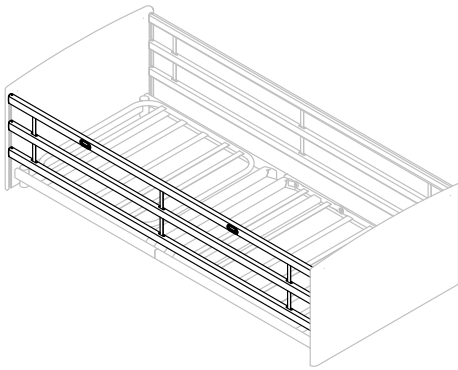

| | | |
|----|---|---|
| en | Side rail for Invacare® NordBed™ User Manual | 1 |
| da | Sengehest til Invacare® NordBed™ Brugsanvisning..... | 2 |
| it | Sponda laterale per Invacare® NordBed™ Manuale d'uso | 3 |
| nl | Zijhek voor Invacare® NordBed™ Gebruiksaanwijzing | 4 |
| no | Sengegrind til Invacare® NordBed™ Bruksanvisning..... | 5 |
| sv | Sänggrind för Invacare® NordBed™ Bruksanvisning..... | 6 |

en

1 Introduction

This manual belongs to an Invacare accessory and is an addition to the manuals of the related Invacare product.

To ensure safety when using this accessory, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.

2 Installing side rails

WARNING!

Risk of personal damage

There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.

- Pay attention to your fingers.
- Follow instructions carefully.
- After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

2.1 Installing the Freya / Embla side rail


WARNING!

Risk of entrapment / suffocation

The user can get trapped or fall out of the bed if the side rail is not correctly mounted or damaged.

- Make sure that all gliding shoes are correctly guided into the bed end guideways.
- Make sure that all side rail bars are above the locking pin and properly locked.
- Make sure that the straps between the bars are not damaged or loose.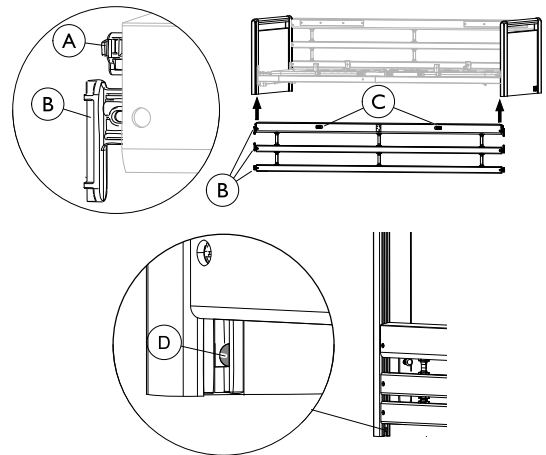

Mounting

1. Lift the upper bar in one end of the side rail. The release buttons © must face up/outwards.
2. Press in the locking pin Ⓐ in the end of the side rail.
3. Guide the three gliding shoes Ⓑ (in the end of the side rail bars) up into the bed end guideways, until they have all passed the locking pin Ⓓ.
4. Repeat steps 2 and 3 to mount the other side of the side rail.

Dismounting

1. Lower the side rail.
2. Lift the lower bar, in one end, to make the locking pin Ⓓ visible and press it in with a pointed object (e.g. a pencil).
3. Lower all three bars, until they are out of the guideways.
4. Repeat steps 2 to 3 to dismount the other end of the side rail.


This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.


Yes, you can.®

3 Operating the Freya / Embla side rail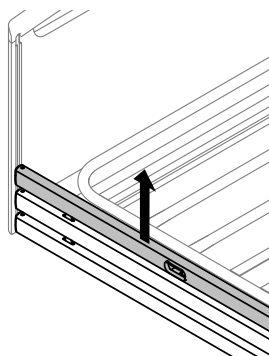


Lock / Upper position

Pull up the top side rail bar, until the locking pin locks with an audible click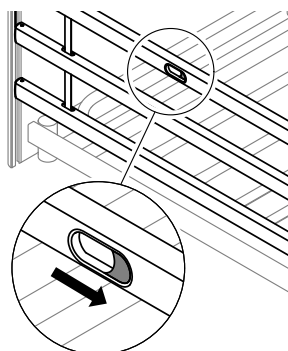

Release

Lift the top side rail bar and press the two locking rings together.

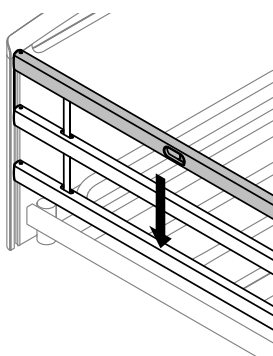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©2019 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare Limited
Invacare Rea AB