
# Invacare® Torill NordBed™

Invacare® Torill NordBed™ Long, Invacare® Torill NordBed™ Short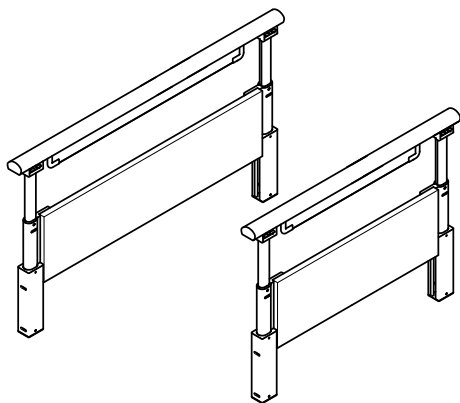

en	Side rail for Invacare® NordBed™ User Manual .....	1
da	Sengehest til Invacare® NordBed™ Brugsanvisning.....	3
it	Sponda laterale per Invacare® NordBed™ Manuale d'uso .....	4
nl	Zijhek voor Invacare® NordBed™ Gebruiksaanwijzing .....	6
no	Sengegrind til Invacare® NordBed™ Bruksanvisning.....	8
sv	Sänggrind för Invacare® NordBed™ Bruksanvisning.....	10

## 1 Introduction

This manual belongs to an Invacare accessory and is an addition to the manuals of the related Invacare product.

To ensure safety when using this accessory, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.

## 2 Installing side rails

### WARNING!

#### Risk of personal damage

There is a risk of entrapment or squeezing, while assembling or disassembling the side rail.

- Pay attention to your fingers.
- Follow instructions carefully.
- After each assembly, check that all fittings are properly tightened and that the side rail has the correct function.

### WARNING!

#### Risk of entrapment / suffocation

There's a risk of entrapment or suffocation between side rail and bed end.

- Never install the short and the long version of the side rail on the same bed.

One side rail per side of bed

- Always make sure that the side rail is mounted close to the head end of the bed and the distance between the bed end and the side rails top bar is more than 3 cm and less than 6 cm.

Two side rails per side of bed

- Always make sure that one side rail is mounted close to the head end and the other close to the foot end of the bed.
- Always make sure that the distance between the bed ends and the side rails top bar is more than 3 cm and less than 6 cm.
- For long side rails, additionally make sure that the middle gap between the two side rails top bars is less than 6 cm.

### Mounting

Depending on the individual need of the end-user only one or two side rails of the same length can be mounted on each side of the bed.

The installation of the long and the short version of the Torill side rail is similar.


This manual MUST be given to the user of the product.  
BEFORE using this product, this manual MUST be read and saved for future reference.


Yes, you can.®

1. Raise the side rail to its high position and remove the two bolts and washers on the brackets.
- 2.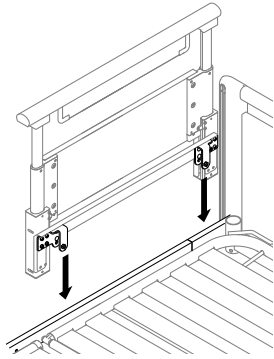

Place both brackets of the side rail on the side tube of the mattress support platform.

3. Position the side rail with a distance of 3–6 cm between the top bar and the bed end.
- 4.

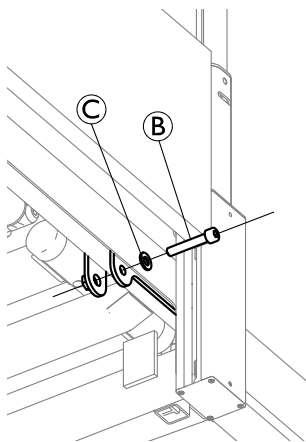Insert the bolt (B) with washer (C) underneath the side tube of the mattress support platform and firmly tighten with a 6 mm Allen key on both brackets of the side rail.

5. Repeat the procedure at the head end on the other side of the bed or at the foot end on the same side of the bed, depending on the side rail version used, desired configuration and availability of additional side rails.

#### Dismount

1. Untighten the bolts and remove on both side rail brackets.
2. Lift the side rail of the side tube.

### 3 Operating the Torill side rail

#### WARNING! Risk of falling

The middle position of the side rail does not provide protection against falling out of the bed.

- Only the highest position is intended to reduce the risk for the end-user of unintentionally falling out of the bed.
- The middle position must only be used to provide support when the end-user is moving in or out of the bed.


#### Raising the side rail

#### IMPORTANT!


Do not push the release buttons when moving the side rail up.

- 1.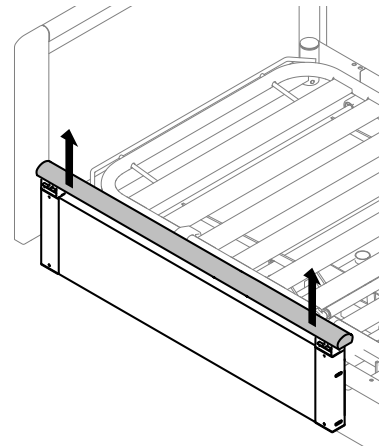

Pull the top bar upwards with both hands near the left and right end.

- 2.

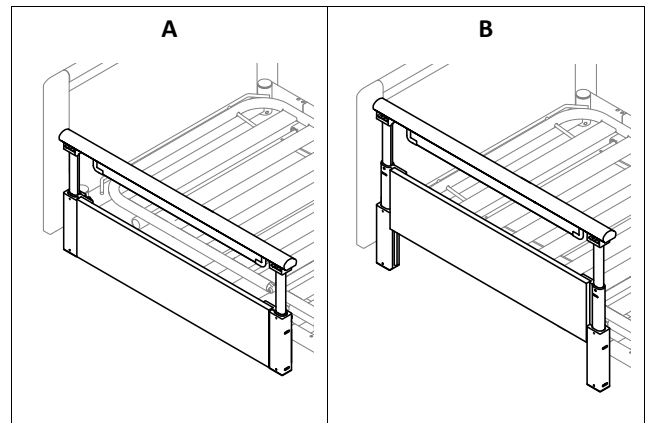To raise the side rail to the middle position (A), stop pulling upwards after a first audible click.

To raise the side rail to the high position (B), pull upwards to the upper stop.


3. Push the top bar downwards to ensure it is properly locked.

If the side rail does not lock properly, perform the following steps to restore proper function.

- Pull up the top bar upwards to its stop, without pushing the release buttons.
- Lower the side rail to its lowest position as described below.

#### Lowering the side rail

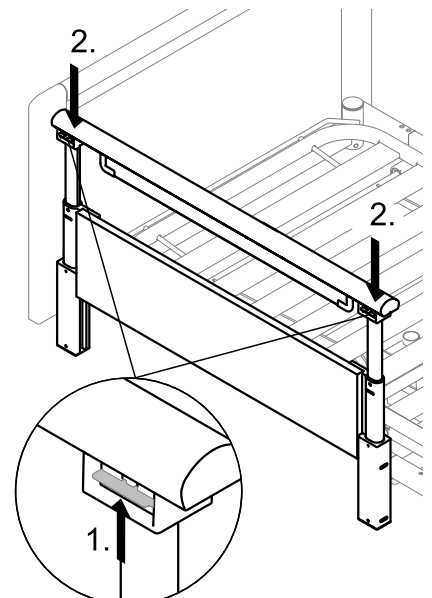1. Push up and hold the two release buttons.
2. To lower the side rail to the middle position, push the top bar downwards and let go the two release buttons after passing the upper locking position.  
To lower the side rail to the lowest position, push the top bar downwards to the lower stop, without releasing the two release buttons.

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©2019 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare Limited
Invacare Rea AB