

PARTS MANUAL



Standard, Extra Wide & Paediatric



Invacare UK Ltd.
Pencoed Technology Park
Pencoed
Bridgend
CF35 5HZ
Tel: 01656 776222
Fax: 01656 776220
E Mail: ordersuk@invacare.com

Invacare Ireland
Unit 5, Seatown Business Campus
Seatown Road
Swords
County Dublin
Ireland
Tel: + 353 1 8107084
Fax: + 353 1 8107085
E Mail: ireland@invacare.com
Web: www.invacare.ie



20100726

TABLE OF CONTENTS

Accessories	2
Axles & Axle Components	3
Anti-Tip Levers	4
Armrests	5-6
Flip Back & Locking	5
Adjustable Height	6
Adjustable Height—Paediatric.....	7
Backrests, Push Handles & Stroller Handles	8
Caster Components	9-11
Caster Forks	9
Caster Components—Mounting.....	10
Caster Wheels	11
Footrests, Footplates	12-14
Composite	12
Flip Up	13
Footrests, Footboard—Removable.....	14
Frame Components	15-19
Frame Assemblies—Base.....	15
Frame Assembly—Seat / Tilt, Std.....	16
Frame Growth Kits	17
Frame Assembly—Gas Spring, X-Wide	18
Frame Assemblies - Tilt	19
Front Rigging	20-21
Pop-Up Hangers	20
Pop-up Elevating Legrest	21
Wheel Locks	22
Wheel Assemblies & Options	23

Terms & Conditions

Warranty Terms & Conditions

Standard INVACARE Terms

This is to certify that your product is warranted by **INVACARE** Ltd. for a period of 1 year for parts and 5 years for frames.

1. Only **INVACARE** products purchased at full price are warranted against defective workmanship and materials.
2. If a defect or fault is discovered the **INVACARE** representative from whom the product was obtained should be notified immediately.
3. The manufacturer will not accept responsibility for damage caused by misuse or non-observance of the instructions set out in the user manual.
4. During the period of the warranty any parts that have become defective, due to faulty workmanship or materials, will be renewed or repaired without charge by the **INVACARE** representative.
5. The warranty will be forfeited should any unauthorised alteration be made to the equipment.
6. The purchaser's statutory rights under the Consumer Protection Act are not affected.

Limitation Of Liability

This warranty does not extend to the consequential costs from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

- Natural wear and tear.
- Inappropriate or incorrect use.
- Defective assembly or setting-up by the purchaser or third parties.
- Defective or neglectful treatment.
- Use of unsuitable spares.


ACCESSORIES



1



2



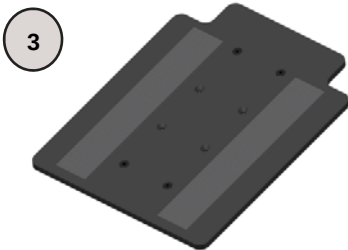


3




4

Protects and Pads Chair
Frame
(Velcro Attachment)

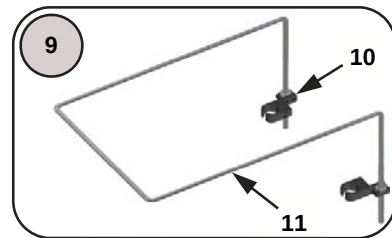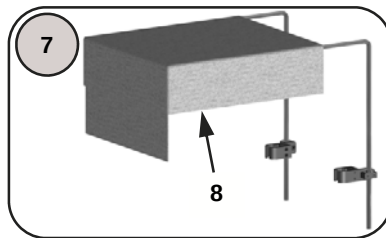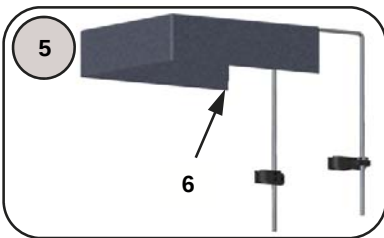


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	FBP	Back Pack	1 ea.	6	10710	Canopy Fabric	1 ea.
2	FMNP	Medical Necessities Bag	1 ea.	7	10713	Canopy Fabric/Visor	1 ea.
3	10910	Seat Pan—Fixed	1 ea.	8	10714	Canopy Fabric/Visor, Frame & Clamps	1 ea.
	10909	Seat Pan—Quick Release	1 ea.	9	10715	Canopy Frame/Clamps	1 ea.
4	10911-2	Frame Pads	1 pr.	10	10716	Canopy Hardware	1 ea.
5	10712	Canopy Fabric/Frame & Clamps	1 ea.	11	10717	Canopy Frame	1 ea..

AXLE COMPONENTS

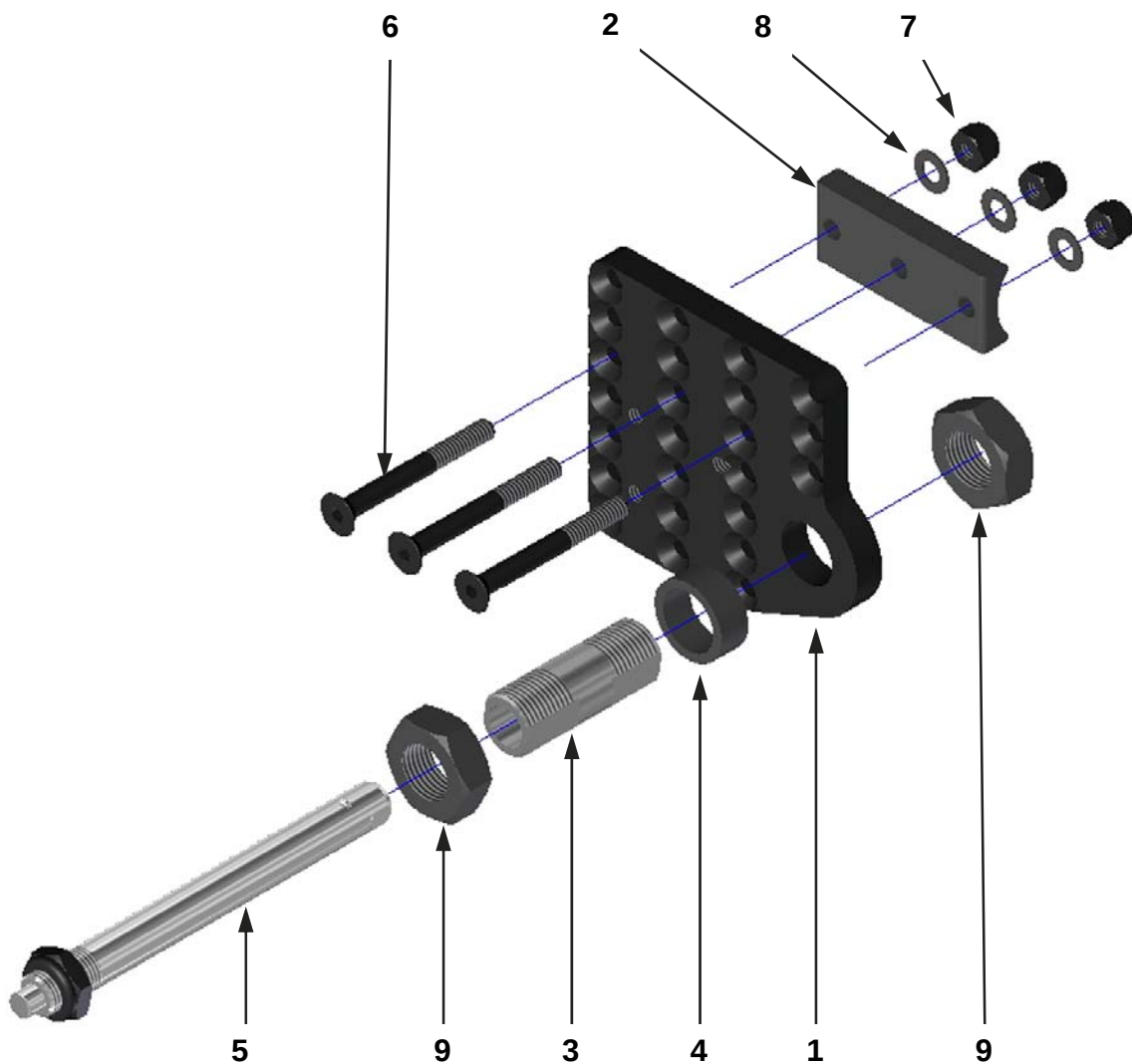

No.	Part #	Description	Qty
1	17005	Axle Plate	1 ea.
2	17006	Axle Plate Spacer	1 ea.
3	2003	Double Lock Axle Receiver	1 ea.
4	1999A	Axle Spacer	1 ea.
5	4001	Quick Release Axle	1 ea.
6	9052	Screw, 1/4-20 X 2-1/4" Flat Head	1 pr.
7	9072	Locknut, 1/4-20 Nylon	1 ea.
8	9074	Washer, 1/4" Flat	1 ea.
9	9092	Jam Nut, 3/4-16 Hex	1 ea.


ANTI-TIP ASSEMBLIES—Flip Up

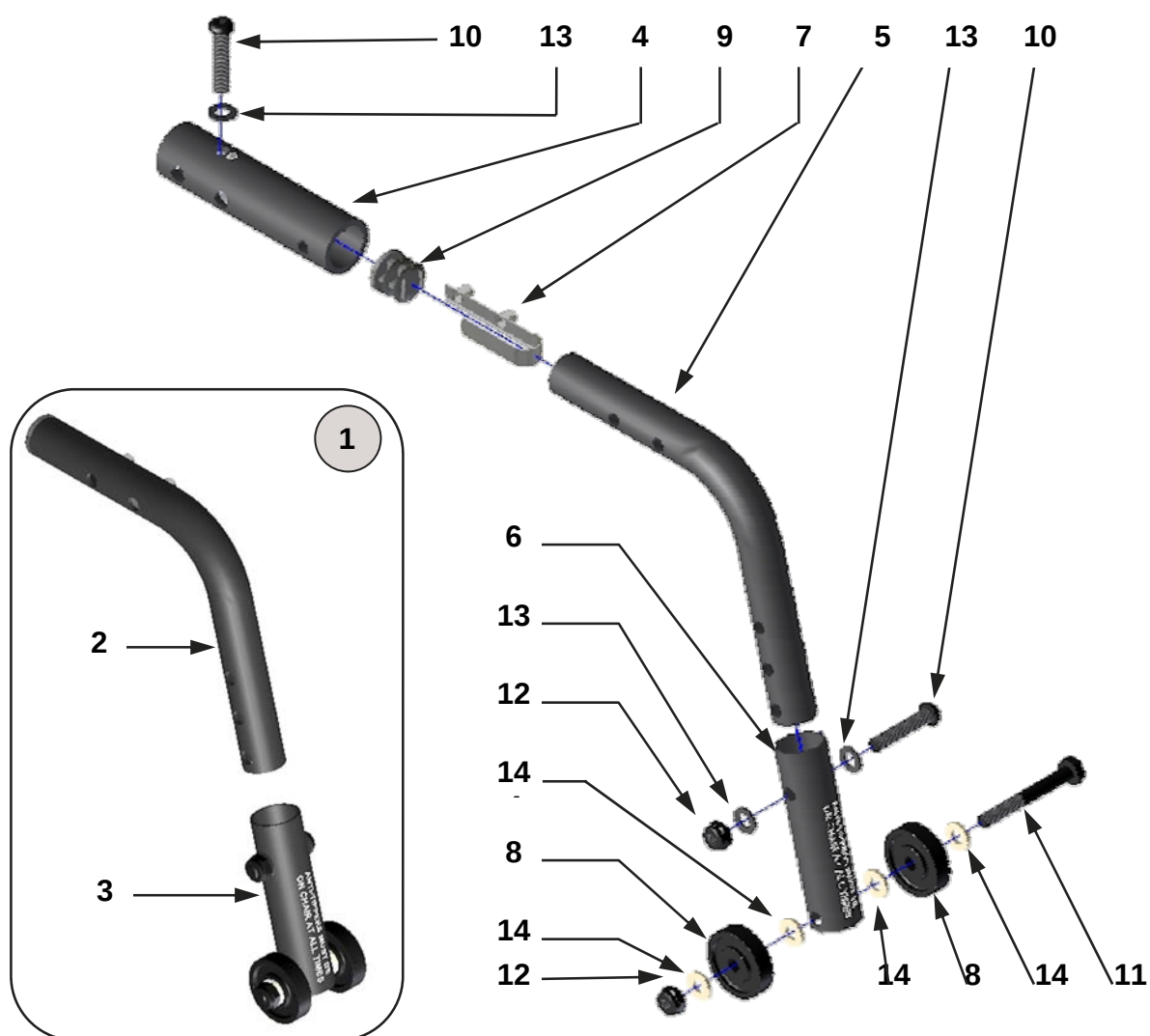

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	17000-L	Anti-Tip Assembly—Complete, Lt	1 ea.	5	17003	Anti-Tip Lever Tube	1 ea.
	17000-R	Anti-Tip Assembly—Complete, Rt	1 ea.	6	8007	Anti-Tip Extension	1 ea.
	17000-2	Anti-Tip Assembly—Complete	1 pr.	7	17004	Dual Snap Button	1 ea.
2	17001-L	Anti-Tip Tube Assembly—Left	1 ea.	8	9009A	Roller	1 ea.
	17001-R	Anti-Tip Tube Assembly—Right	1 ea.	9	3099	Tube Plug, 3/4"	1 ea.
	17001-2	Anti-Tip Tube Assembly	1 pr.	10	9034	Screw, 1/4-20 X 1-1/4" Button Hd	1 ea.
3	8001-L	Anti-Tip Roller Assembly—Left	1 ea.	11	9023	Screw, 1/4-20 X 2" Hex Head	1 ea.
	8001-R	Anti-Tip Roller Assembly—Right	1 ea.	12	9073	Locknut, 1/4-20 Nylon—Thin	1 ea.
	8001-2	Anti-Tip Roller Assembly	1 pr.	13	9074	Washer, 1/4" Flat	1 ea.
4	17002	Anti-Tip Housing Assembly	1 ea.	14	9010	Washer, 1/4" Nylon	1 ea.





ARMRESTS—Flip Back

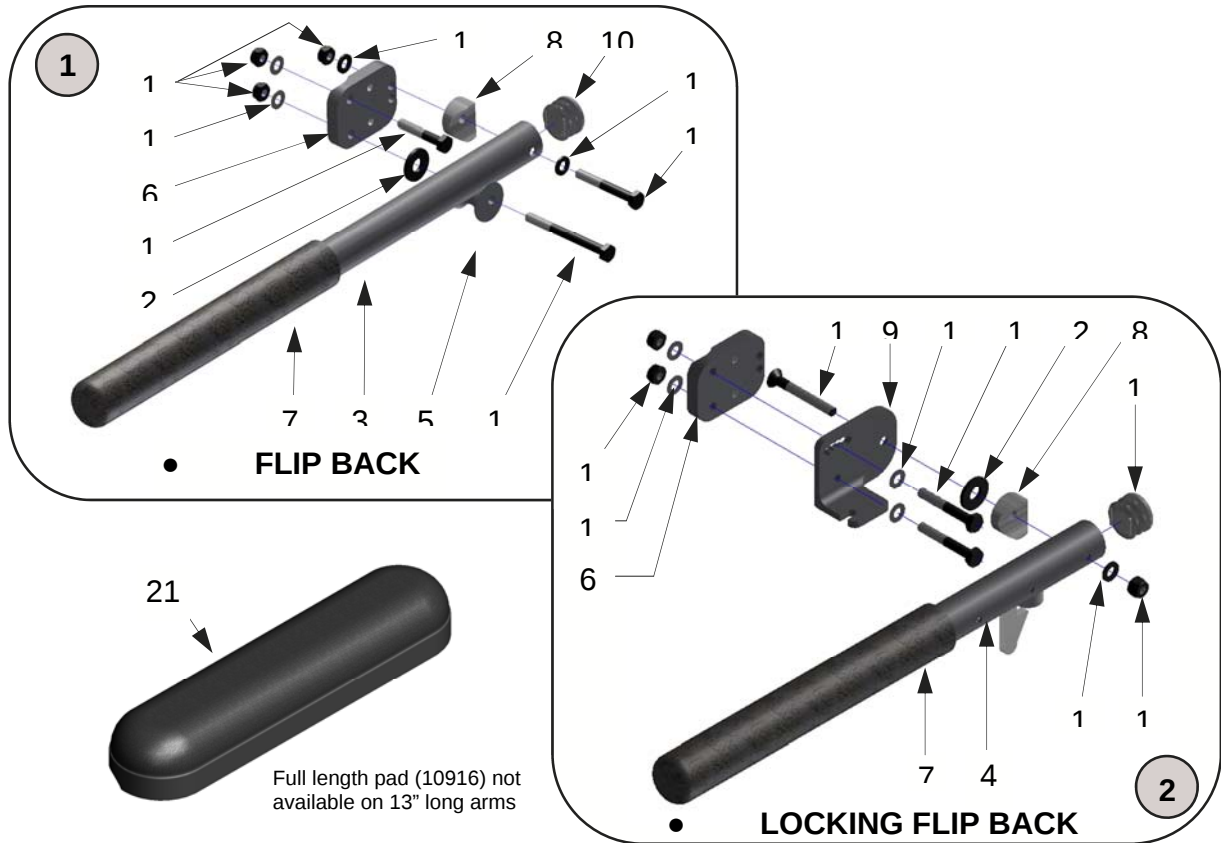

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	10721-L	Flip Back Armrest Assy—16", Lt	1 ea.		5022	Foam Pad—9 1/2"	1 ea.
	10721-R	Flip Back Armrest Assy—16", Rt	1 ea.	8	9005	Plastic Saddle, 1" for 5/16" Screw	1 ea.
	10721-2	Flip Back Armrest Assemblies—16"	1 pr.	9	6004-L	Adjustment Plate—Left	1 ea.
	10721-PL	Flip Back Armrest Assy—13", Lt	1 ea.		6004-R	Adjustment Plate—Right	1 ea.
	10721-PR	Flip Back Armrest Assy—13"- Rt	1 ea.		6004-2	Adjustment Plates—Right & Left	1 pr.
	10721-P-2	Flip Back Armrest Assemblies—13"	1 pr.	10	9000	Tube Plug, 1"	1 ea.
2	10751-L	Locking Armrest Assy—16", Lt	1 ea.	11	9021	Screw, 1/4-20 x 1 1/2" Hex Head	1 ea.
	10751-R	Locking Armrest Assy—16", Rt	1 ea.	12	9022	Screw, 1/4-20 x 1 3/4" Hex Head	1 ea.
	10751-2	Locking Armrest Assemblies—16"	1 pr.	13	9023	Screw, 1/4-20 x 2" Hex Head	1 ea.
	10751-PL	Locking Armrest Assy—13", Lt	1 ea.	14	9026	Screw, 1/4-20 x 2 3/4" Hex Head	1 ea.
	10751-PR	Locking Armrest Assy—13", Rt	1 ea.	15	9051	Screw, 1/4-20 x 2" Flat Head	1 ea.
	10751-P	Locking Armrest Assemblies—13"	1 pr.	16	9072	Locknut, 1/4-20 Nylon	1 ea.
3	10722	Flip Back Tube (no hardware) - 16"	1 ea.	17	9073	Locknut, 1/4-20 Nylon—Thin	1 ea.
	10722-P	Flip Back Tube (no hardware) - 13"	1 ea.	18	9074	Washer, 1/4" Flat	1 ea.
4	10754	Locking Arm Tube Assembly—16"	1 ea.	19	9075	Washer, 1/4" Flat—Thin	1 ea.
	10754-P	Locking Arm Tube Assembly—13"	1 ea.	20	9089	Washer, 5/16" Flat	1 ea.
5	6012	Flip Back Armrest Barrel	1 ea.	21	10913	Desk Length Arm Pad—9 1/4" Lg	1 ea.
6	6014	Mounting Plate	1 ea.		10916	Full Length Arm Pad—13 1/4" Lg	1 ea.
7	5021	Foam Pad—4 1/2"	1 ea.				



ARMRESTS—Adjustable Height

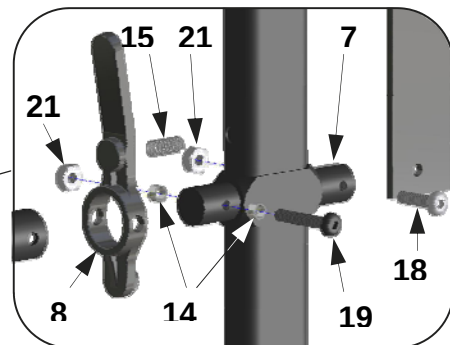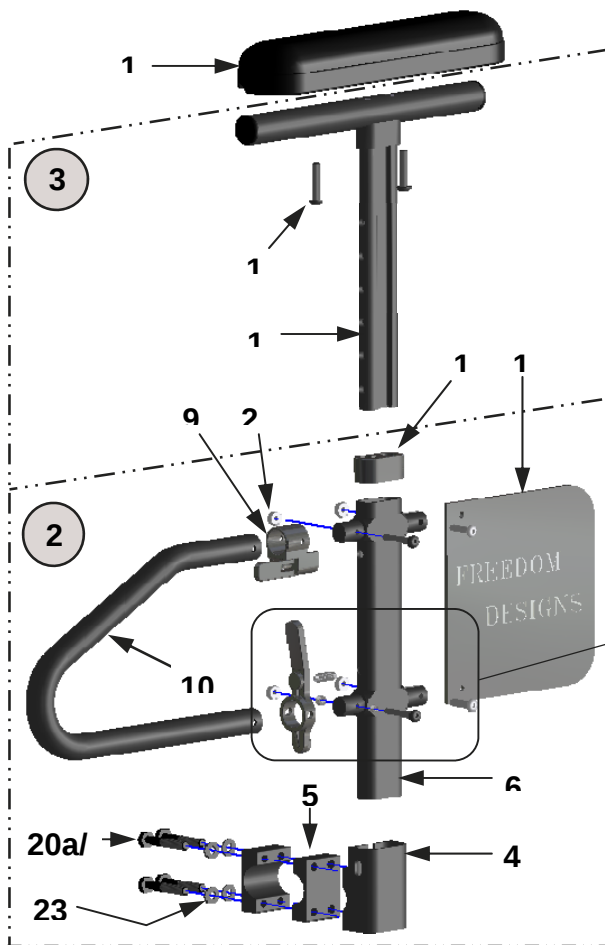

STANDARD SEAT WIDTH REQUIRES (1) INNER CLAMP (ITEM 5) AND USES 1-1/4" LONG MOUNTING SCREW (ITEM 20a)
EXTENDED SEAT WIDTH REQUIRES (2) INNER CLAMPS AS SHOWN AND USES 1-1/2" LONG MOUNTING SCREWS (ITEM 20b)

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	10756-L	T-Arm Assembly—Complete, Left	1 ea.	10	10766-STD	Front Tube—Standard	1 ea.
	10756-R	T-Arm Assembly—Complete, Right	1 ea.		10766-SH	Front Tube—Short	1 ea.
	10756-2	T-Arm Assemblies—Comp., Rt & Lt	1 pr.	11	10767	Panel	1 ea.
2	10757-L	Lower T-Arm Assembly—Left	1 ea.	12	10768	Receiver Tube Cap	1 ea.
	10757-R	Lower T-Arm Assembly—Right	1 ea.	13	10769	Height Adjust Post	1 ea.
	10757-2	Lower T-Arm Assemblies—Lt & Rt	1 pr.	14	9039	Spacer Bushing	1 ea.
3	10758-L	Upper T-Arm Assembly—Left	1 ea.	15	9040	Spring	1 ea.
	10758-R	Upper T-Arm Assembly—Right	1 ea.	16	10913	Desk Style Arm Pad 9 1/4" Long	1 ea.
	10758-2	Upper T-Arm Assemblies—Lt & Rt	1 pr.		10916	Full Length Arm Pad 13 1/4" Long	1 ea.
4	10759	Receiver Socket	1 ea.	17	9016	Screw, 10-32 X 1 1/4" Pan Head	1 ea.
5	10760	Receiver Frame Clamp	1 ea.	18	9057	Screw, 10-32 X 3/4" Button Head	1 ea.
6	10761	Receiver Tube	1 ea.	19	9070	Screw, 10-32 X 1 1/4" Button Head	1 ea.
7	10763	Receiver Tube Clamp	1 ea.	20a	9020	Screw, 1/4-20 X 1 1/4" Hex Head	1 ea.
8	10764	Frame Release Lever	1 ea.	20b	9021	Screw, 1/4-20 X 1 1/2" Hex Head	1 ea.
9	10765-R	Height Release Assembly—Right	1 ea.	21	9086	Locknut, 10-32 Nylon	1 ea.
	10765-L	Height Release Assembly—Left	1 ea.	22	9074	Washer, 1/4" Flat	1 ea.




ARMRESTS—Adjustable Height, Paediatric

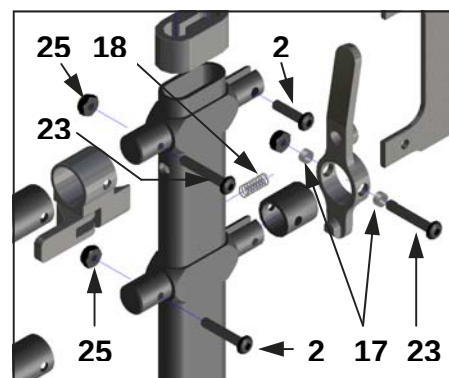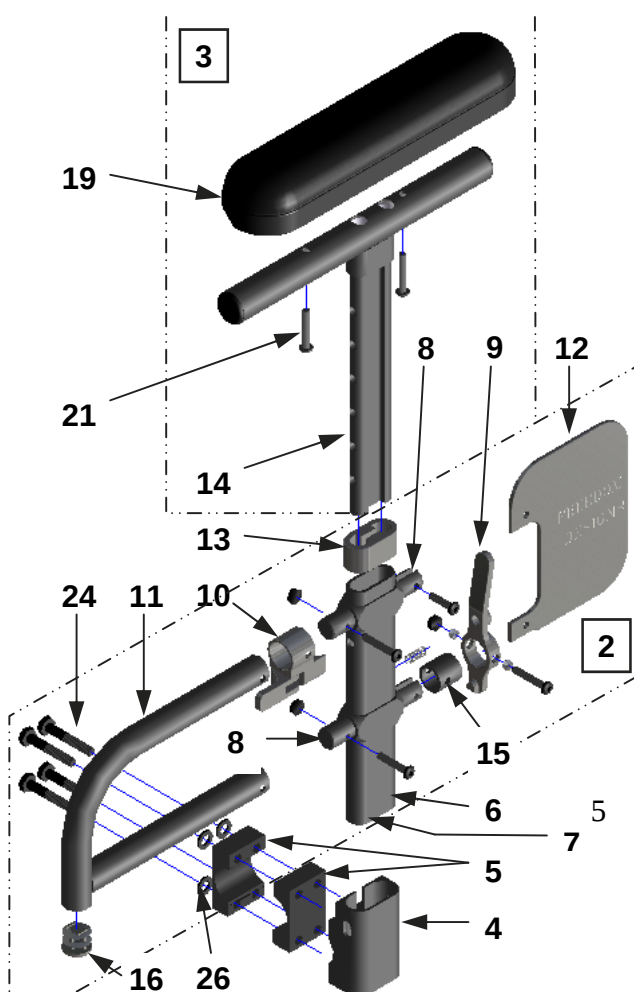

STANDARD SEAT WIDTH REQUIRES (1) INNER CLAMP (ITEM 5) AND USES 1-1/4" LONG MOUNTING SCREW (ITEM 21a)
EXTENDED SEAT WIDTH REQUIRES (2) INNER CLAMPS AS SHOWN AND USES 1-1/2" LONG MOUNTING SCREWS (ITEM 21b)


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	10780-L	T-Arm Assembly, Ped. Complete—Lt.	1 ea.	11	10766-P	Front Tube—Paediatric	1 ea.
	10780-R	T-Arm Assembly, Ped. Complete—Rt.	1 ea.	12	10783	Panel, T-Arm	1 ea.
	10780-2	T-Arm Assemblies, Ped. Cmplt.—R & L	1 pr.	13	10768	Receiver Tube Cap, T-Arm	1 ea.
2	10781-L	Lower T-Arm Assembly, Ped.—Lt.	1 ea.	14	10769-P	Height Adjust Post, T-Arm	1 ea.
	10781-R	Lower T-Arm Assembly, Ped.—Rt.	1 ea.	15	10784	Spacer Tube, T-Arm	1 ea.
	10781-2	Lower T-Arm Assemblies, Ped.—Lt & Rt	1 pr.	16	3099	Tube Plug—3/4"	1 ea.
3	10782-L	Upper T-Arm Assembly, Ped.—Left	1 ea.	17	9039	Spacer Bushing, T-Arm	1 ea.
	10782-R	Upper T-Arm Assembly, Ped.—Right	1 ea.	18	9040	Spring, T-Arm	1 ea.
	10782-2	Upper T-Arm Assemblies, Ped.—Lt & Rt	1 pr.	19	10913	Desk Style Arm Pad 9 1/4" Long	1 ea.
4	10759	Receiver Socket, T-Arm	1 ea.	20	9102	Screw, 10-32 x 1" Button Head	1 ea.
5	10760	Receiver Frame Clamp, T-Arm	1 ea.	21	9016	Screw, 10-32 x 1 1/4" Phillips Pan Head	1 ea.
6	10761-P	Receiver Tube, T-Arm	1 ea.	22	9057	Screw, 10-32 x 3/4" Button Head	1 ea.
7	10763	Receiver Tube Clamp, T-Arm	1 ea.	23	9070	Screw, 10-32 x 1 1/4" Button Head	1 ea.
8	10764	Frame Release Lever, T-Arm	1 ea.	24a	9020	Screw, 1/4-20 x 1 1/4" Hex Head	1 ea.
9	10765-L	Height Release Assembly—Left	1 ea.	24b	9021	Screw, 1/4-20 x 1 1/2" Hex Head	1 ea.
	10765-R	Height Release Assembly—Right	1 ea.	25	9086	Locknut, 10-32 Nylon	1 ea.
	10765-2	Height Release Assemblies—Lt. & Rt.	1 pr.	26	9074	Washer, 1/4" Flat	1 ea.

BACK CANES & STROLLER HANDLES

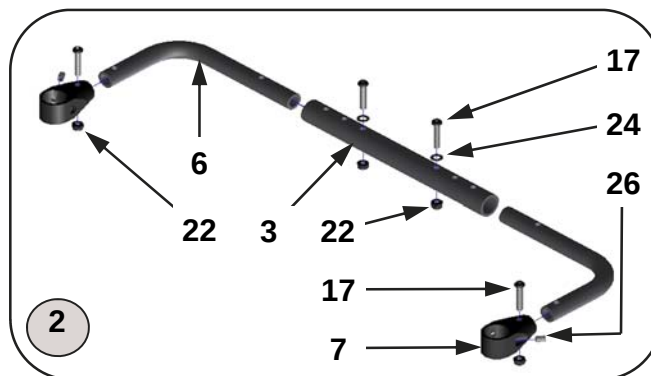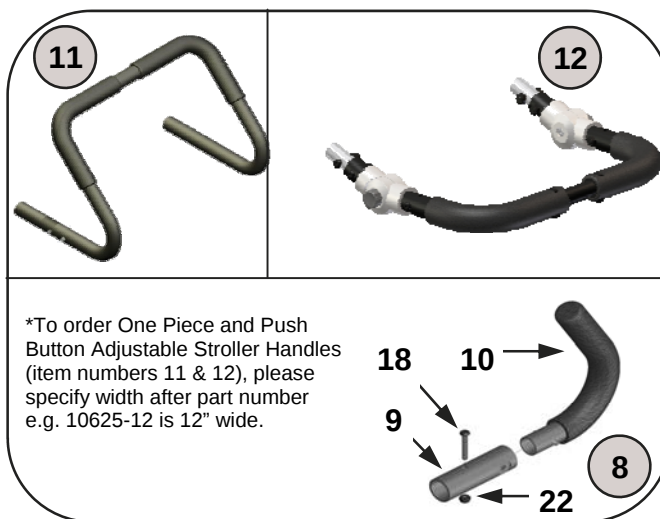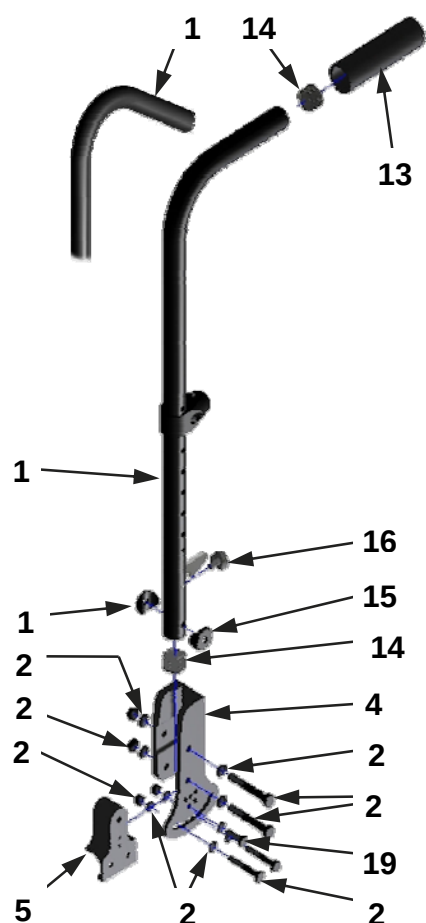


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	10610-2	Backpost Tubes—Short	1 pr.	9	10622	Adapter Tube	1 ea.
	10611-2	Backpost Tubes—Medium	1 pr.	10	10621	Stroller Handle	1 ea.
	10612-2	Backpost Tubes—Tall	1 pr.	11	*10623	1 Pc. Push Handle Extension. Q. R.	1 ea.
1a	13203-2	Backposts, Ergo—Short	1 pr.	12	*10625	Push Button Adj. Stroller Handle	1 ea.
	13204-2	Backposts, Ergo—Medium	1 pr.	13	1998	Foam Grip	1 ea.
	13205-2	Backposts, Ergo—Tall	1 pr.	14	9000	Tube Plug, 1"	1 ea.
2	17050-N	Rigidising Assembly—Narrow	1 ea.	15	9005	Plastic Saddle, 1" for 5/16" Screw	1 ea.
	17050-M	Rigidising Assembly—Medium	1 ea.	16	17010	Bumper Plug	1 ea.
	17050-W	Rigidising Assembly—Wide	1 ea.	17	9034	Screw, 1/4-20 X 1-1/4" Button Head	1 ea.
3	17051-N	Rigidising Centre Tube—Narrow	1 ea.	18	9035	Screw, 1/4-20 X 1-1/2" Button Head	1 ea.
	17051-M	Rigidising Centre Tube - Medium	1 ea.	19	9102	Screw, 1/4-20 X 7/8" Hex Head	1 ea.
	17051-W	Rigidising Centre Tube—Wide	1 ea.	20	9022	Screw, 1/4-20 X 1-3/4" Hex Head	1 ea.
4	17052-R	Backpost Receiver—Right	1 ea.	21	9081	Screw, 5/16-18 X 2-1/4" Hex Head	1 ea.
	17052-L	Backpost Receiver—Left	1 ea.	22	9073	Locknut, 1/4-20 Nylon—Thin	1 ea.
5	17053	Backpost Clamp	1 ea.	23	9088	Locknut, 5/16-18 Nylon—Thin	1 ea.
6	17054	Rigidizer Side Tube	1 ea.	24	9074	Washer, 1/4" Flat	1 ea.
7	16049	Rigidising Backrest Bar Mount Fitting	1 ea.	25	9090	Washer, 5/16" Flat—Thin	1 ea.
8	10620	Stroller Handle Assembly	1 ea.	26	9103	Set Screw, 1/4-20 X 3/8"	1 ea.

CASTOR COMPONENTS—Forks

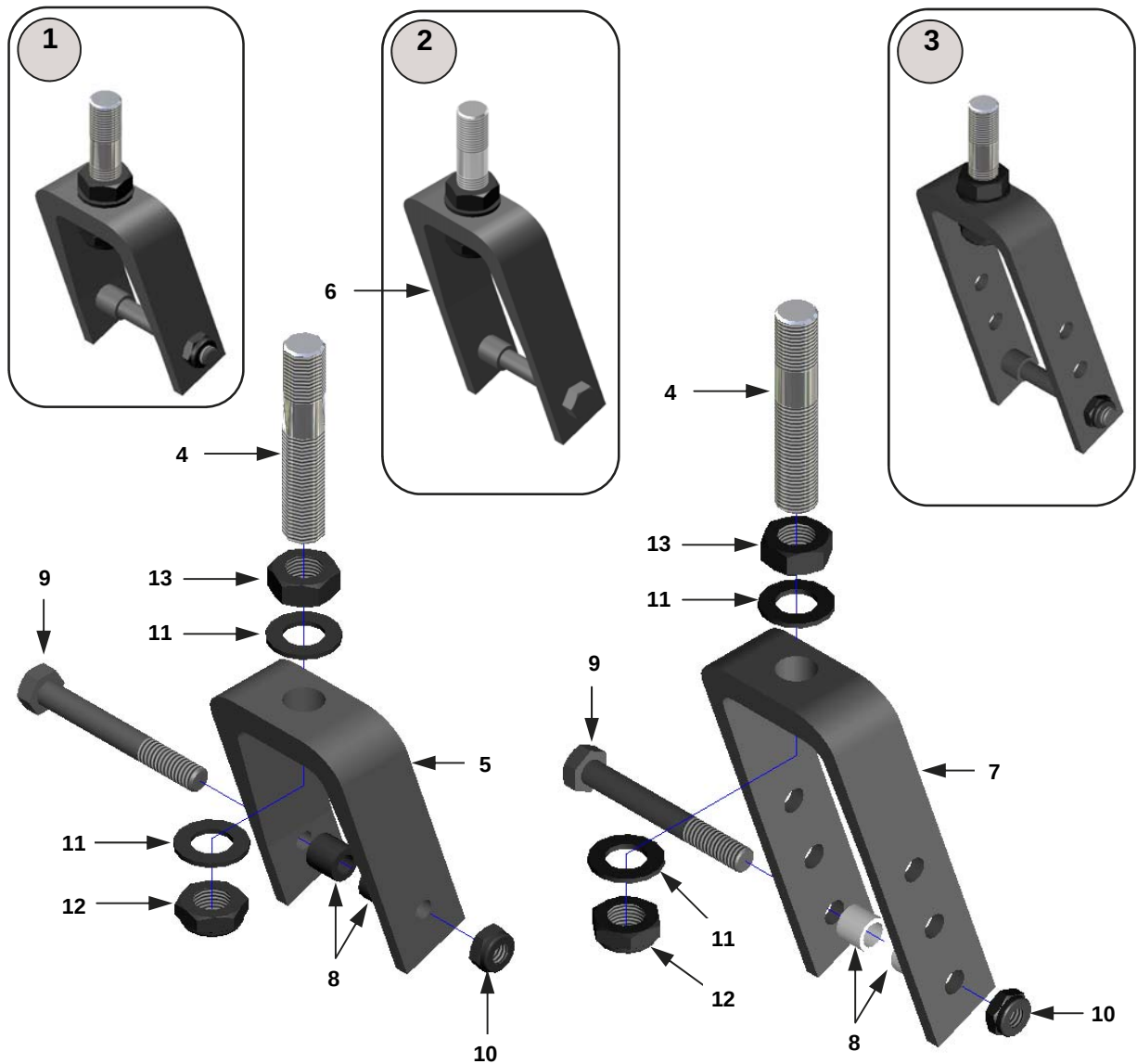


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	10044-1	3" Castor Fork Assembly	1 ea.	6	10046-1	4" Castor Fork	1 ea.
	10044-2	3" Castor Fork Assemblies	1 pr.	7	2011A	5", 6" & 7" Castor Fork	1 ea.
2	10052-1	4" Castor Fork Assembly	1 ea.	8	2013	Castor Spacer (except 6" x 2")	1 ea.
	10052-2	4" Castor Fork Assemblies	1 pr.	9	9082	Screw, 5/16-18 X 2 1/2" Hex Head	1 ea.
3	10053-1	5", 6" & 7" Castor Fork Assembly	1 ea.	10	9088	Locknut, 5/16-18 Nylon—Thin	1 ea.
	10053-2	5", 6" & 7" Castor Fork Assemblies	1 pr.	11	9089	Washer, 1/2" Flat	1 ea.
4	2010A	3-5/8" Castor Stem	1 ea.	12	9085	Locknut, 1/2-20 Nylon - Thin	1 ea.
	2010B	4" Castor Stem	1 ea.	13	9091	Nut, 1/2" Hex Jam	1 ea.
5	2011	3" Castor Fork	1 ea.				

CASTOR COMPONENTS—Mounting

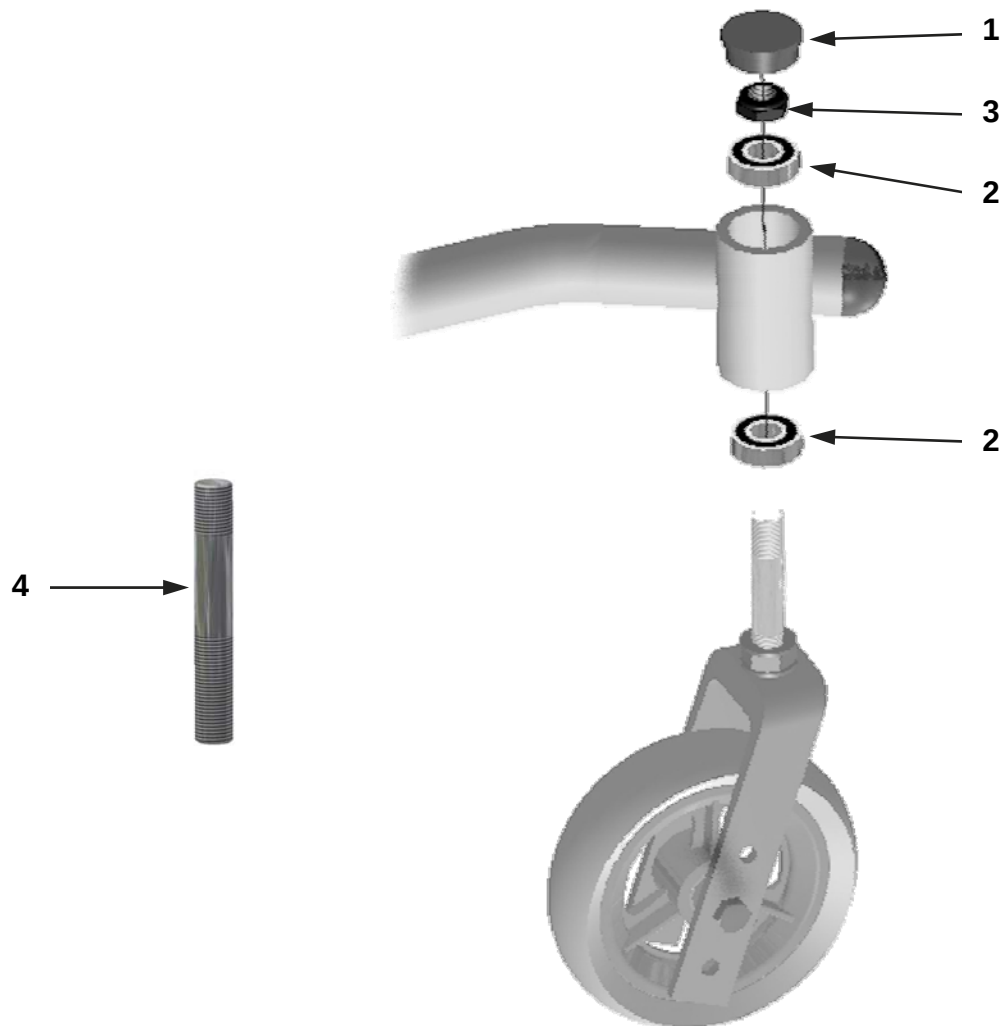


No.	PART #	DESCRIPTION	QTY
1	1720	Cap—Castor Barrel	1 ea.
2	2009	Bearing, 1 1/8" x 1/2"	1 ea.
3	9085	Locknut, 1/2-20 Thin Nylon	1 ea.
4	2010A	Castor Stem, 3-5/8"	1 ea.

CASTOR COMPONENTS – Wheels & Bearings

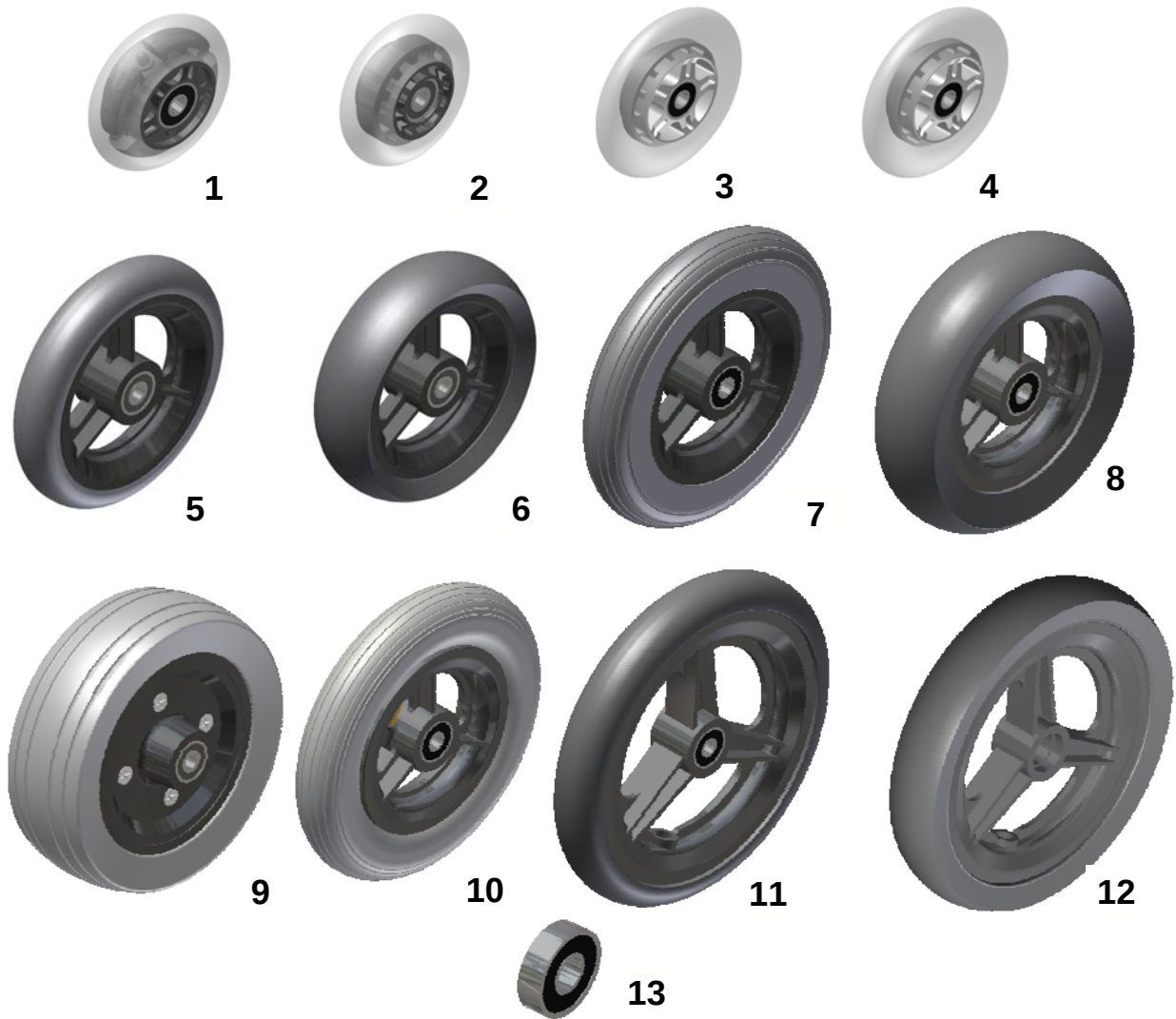


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	10141-1	3" Micro Light Up Castor	1 ea.		10144-2	6" x 1" Poly Castors	1 pr.
	10141-2	3" Micro Light Up Castors	1 pr.	8	10150-1	6" x 1 1/2" Poly Castor	1 ea.
2	10142-1	3" Micro Castor	1 ea.		10150-2	6" x 1 1/2" Poly Castors	1 pr.
	10142-2	3" Micro Castor	1 pr.	9	10151-1	6" x 2" Poly Castor	1 ea.
3	10152-1	4" Micro Castor	1 ea.		10151-2	6" x 2" Poly Castors	1 pr.
	10152-2	4" Micro Castors	1 pr.	10	10145-1	6" x 1" Pneumatic Castor	1 ea.
4	10153-1	4" Micro Light Up Castor	1 ea.		10145-2	6" x 1" Pneumatic Castors	1 pr.
	10153-2	4" Micro Light Up Castors	1 pr.	11	10146-1	7" x 1" Poly Castor	1 ea.
5	10143-1	5" x 1" Poly Castor	1 ea.		10146-2	7" x 1" Poly Castors	1 pr.
	10143-2	5" x 1" Poly Castors	1 pr.	12	10148-1	7" x 1 1/2" Castor	1 ea.
6	10149-1	5" x 1 1/2" Poly Castor	1 ea.		10148-2	7" x 1 1/2" Castors	1 pr.
	10149-2	5" x 1 1/2" Poly Castors	1 pr.	13	2009A	Castor Wheel Bearing	1 ea.
7	10144-1	6" x 1" Poly Castor	1 ea.				

FOOTRESTS—Composite Flip-up Footplate

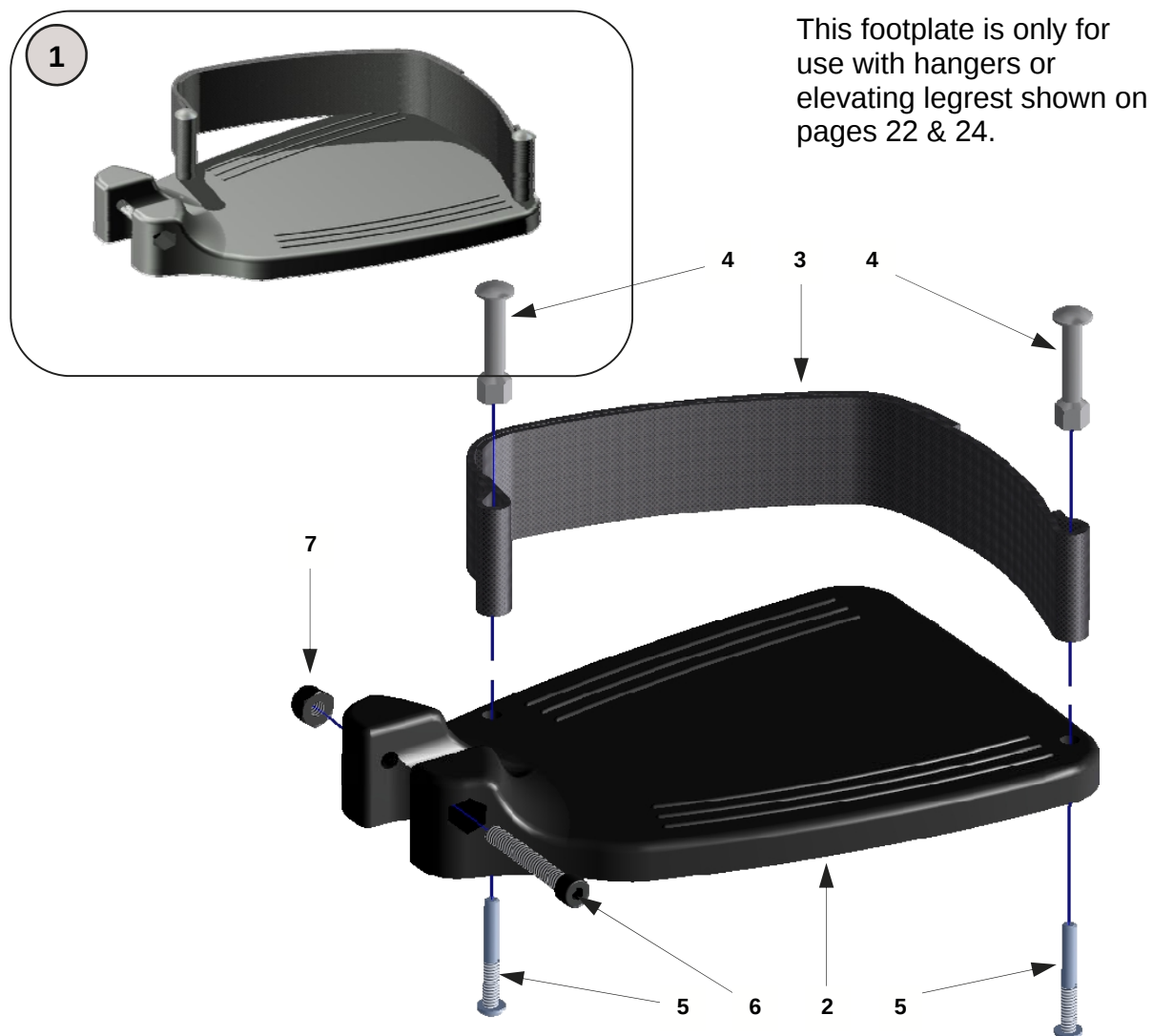


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	20071-L	Footplate Assembly, 12-13", Left	1 ea.		20075-R	Footplate, 12-13", Right	1 ea.
	20071-R	Footplate Assembly, 12-13", Right	1 ea.		20076-L	Footplate, 14-15", Left	1 ea.
	20071	Footplate Assemblies, 12-13"	1 pr.		20076-R	Footplate, 14-15", Right	1 ea.
	20072-L	Footplate Assembly, 14-15", Left	1 ea.		20077-L	Footplate, 16-17", Left	1 ea.
	20072-R	Footplate Assembly, 14-15", Right	1 ea.		20077-R	Footplate, 16-17", Right	1 ea.
	20072	Footplate Assemblies, 14-15"	1 pr.		20078-L	Footplate—18", Left	1 ea.
	20073-L	Footplate Assembly, 16-17", Left	1 ea.		20078-R	Footplate—18", Right	1 ea.
	20073-R	Footplate Assembly, 16-17", Right	1 ea.	3	EE9D	Heel Loops	1 pr.
	20073	Footplate Assemblies, 16-17"	1 pr.	4	9108	Heel Loop Mount Stud	1 ea.
	20074-L	Footplate Assembly—18", Left	1 ea.	5	9109	Heel Loop Mount Screw	1 ea.
	20074-R	Footplate Assembly—18", Right	1 ea.	6	9064	Screw, 1/4-20 x 1 3/4" Socket Head	1 ea.
	20074	Footplate Assemblies—18"	1 pr.	7	9072	Locknut, 1/4-20 Nylon	1 ea.
2	20075-L	Footplate, 12-13", Left	1 ea.				

FOOTRESTS—Flip Up Footplate

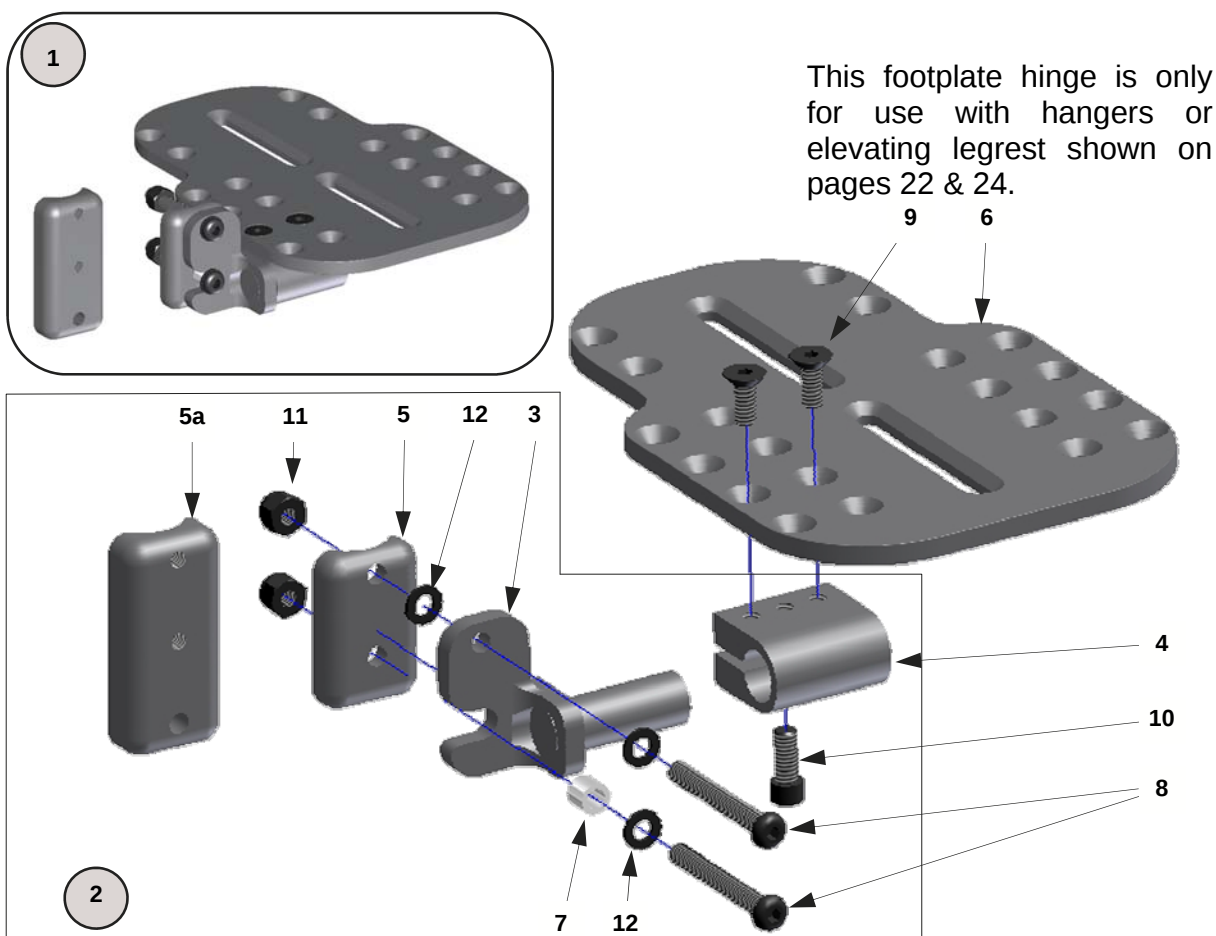


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	20050-L	Flip Up Footplate Assy, 10-11", Lt.	1 ea.		20051-2	Pivot Arm Assemblies	1 pr.
	20050-R	Flip Up Footplate Assy, 10-11", Rt.	1 ea.	3	20065-L	Pivot Arm—Left	1 ea.
	20050-2	Flip Up Footplate Assy, 10-11"	1 pr.		20065-R	Pivot Arm—Right	1 ea.
	20060-L	Flip Up Footplate Assy, 12-13", Lt.	1 ea.	4	20066	Footplate Swivel Clamp	1 ea.
	20060-R	Flip Up Footplate Assy, 12-13", Rt.	1 ea.	5	20067	Hanger Spacer	1 ea.
	20060-2	Flip Up Footplate Assy, 12-13"	1 pr.	5a	20068	Hanger Spacer—Extended	1 ea.
	20061-L	Flip Up Footplate Assy, 14-15", Lt.	1 ea.	6	20040	Footplate—Flip Up Angle Adj., 10-11"	1 ea.
	20061-R	Flip Up Footplate Assy, 14-15", Rt.	1 ea.		20041	Footplate—Flip Up Angle Adj., 12-13"	1 ea.
	20061-2	Flip Up Footplate Assy, 14-15"	1 pr.		20042	Footplate—Flip Up Angle Adj., 14-15"	1 ea.
	20062-L	Flip Up Footplate Assy, 16-17", Lt.	1 ea.		20043	Footplate—Flip Up Angle Adj., 16-17"	1 ea.
	20062-R	Flip Up Footplate Assy, 16-17", Rt.	1 ea.		20044	Footplate—Flip Up Angle Adj., 18"	1 ea.
	20062-2	Flip Up Footplate Assy, 16-17"	1 pr.	7	20069	Footplate Pivot Spacer	1 ea.
	20063-L	Flip Up Footplate Assy, 18", Lt.	1 ea.	8	9037	Screw, 1/4-20 x 2" Button Head	1 ea.
	20063-R	Flip Up Footplate Assy, 18", Rt.	1 ea.	9	9053	Screw, 1/4-20 x 5/8" Flat Head	1 ea.
	20063-2	Flip Up Footplate Assy, 18"	1 pr.	10	9107	Screw, 1/4-20 x 5/8" Socket Head	1 ea.
2	20051-L	Pivot Arm Assembly—Left	1 ea.	11	9072	Locknut, 1/4-20 Nylon	1 ea.
	20051-R	Pivot Arm Assembly—Right	1 ea.	12	9074	Washer, 1/4" Flat	1 ea.

FOOTRESTS—Removable Footboard

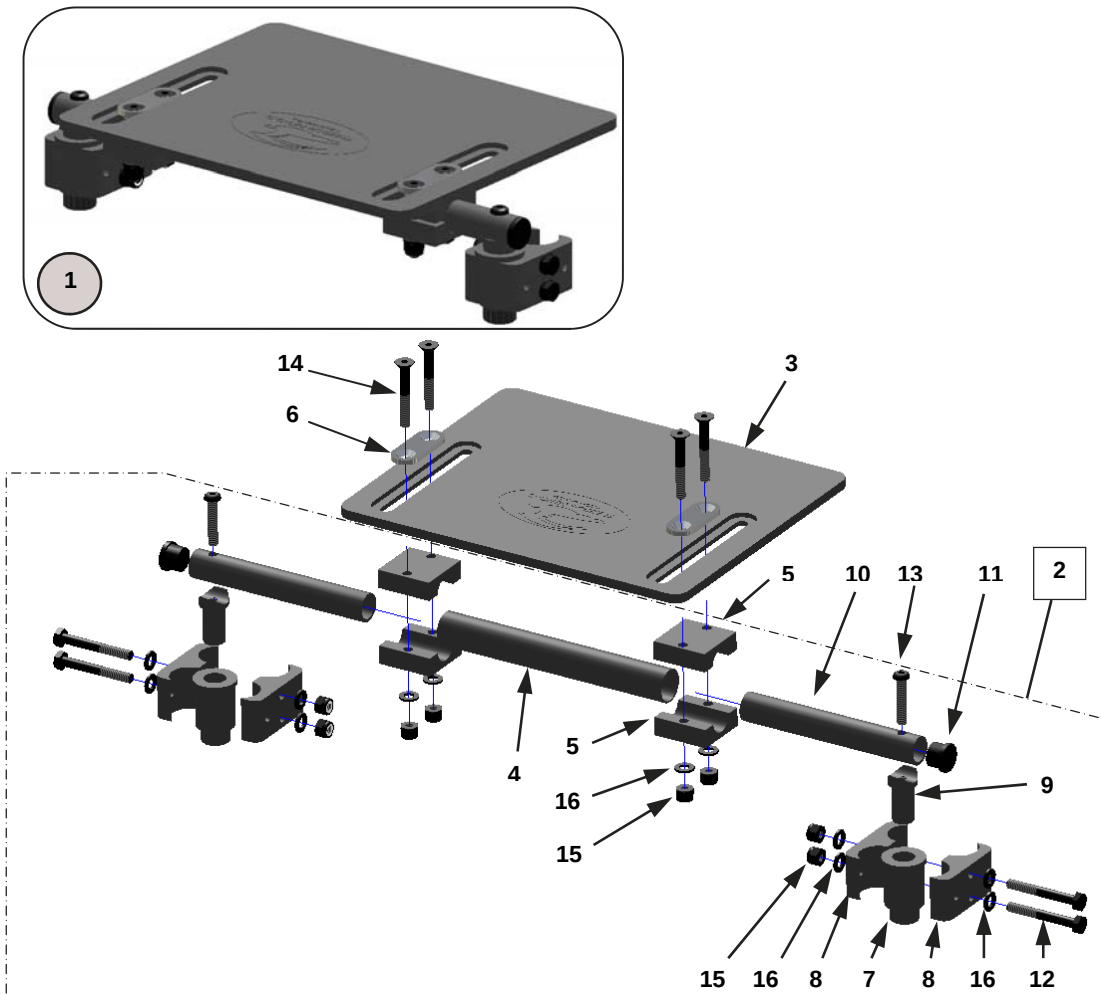


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	10200	Removable Footboard Assy, 10-11"	1 ea.		3033	Spacer Tube, 12-13"	1 ea.
	10201	Removable Footboard Assy, 12-13"	1 ea.		3034	Spacer Tube, 14-15"	1 ea.
	10202	Removable Footboard Assy, 14-15"	1 ea.		3035	Spacer Tube, 16-17"	1 ea.
	10203	Removable Footboard Assy, 16-17"	1 ea.		3036	Spacer Tube, 18"	1 ea.
	10204	Removable Footboard Assy, 18"	1 ea.	5	3027	Footboard Clamp	1 ea.
2	13167	Hardware Assembly, 10-11"	1 set	6	3028	Clamp Plate	1 ea.
	13168	Hardware Assembly, 12-13"	1 set	7	3029	Receiver Assembly	1 ea.
	13169	Hardware Assembly, 14-15"	1 set	8	3024	Frame Clamp	1 ea.
	13170	Hardware Assembly, 16-17"	1 set	9	3025	Locking Insert	1 ea.
	13171	Hardware Assembly, 18"	1 set	10	3026	Adjustment Tube	1 ea.
3	2037	Removable Footboard, 10-11"	1 ea.	11	3099	Tube Plug, 3/4"	1 ea.
	2038	Removable Footboard, 12-13"	1 ea.	12	9023	Screw, 1/4-20 x 2" Hex Head	1 ea.
	2039	Removable Footboard, 14-15"	1 ea.	13	9034	Screw, 1/4-20 x 1 1/4" Button Head	1 ea.
	2040	Removable Footboard, 16-17"	1 ea.	14	9050	Screw, 1/4-20 x 1 3/4" Flat Head	1 ea.
	2041	Removable Footboard, 18"	1 ea.	15	9072	Locknut, 1/4-20 Nylon	1 ea.
4	3032	Spacer Tube, 10-11"	1 ea.	16	9074	Washer, 1/4" Flat	1 ea.

BASE FRAME—Standard & X-Wide

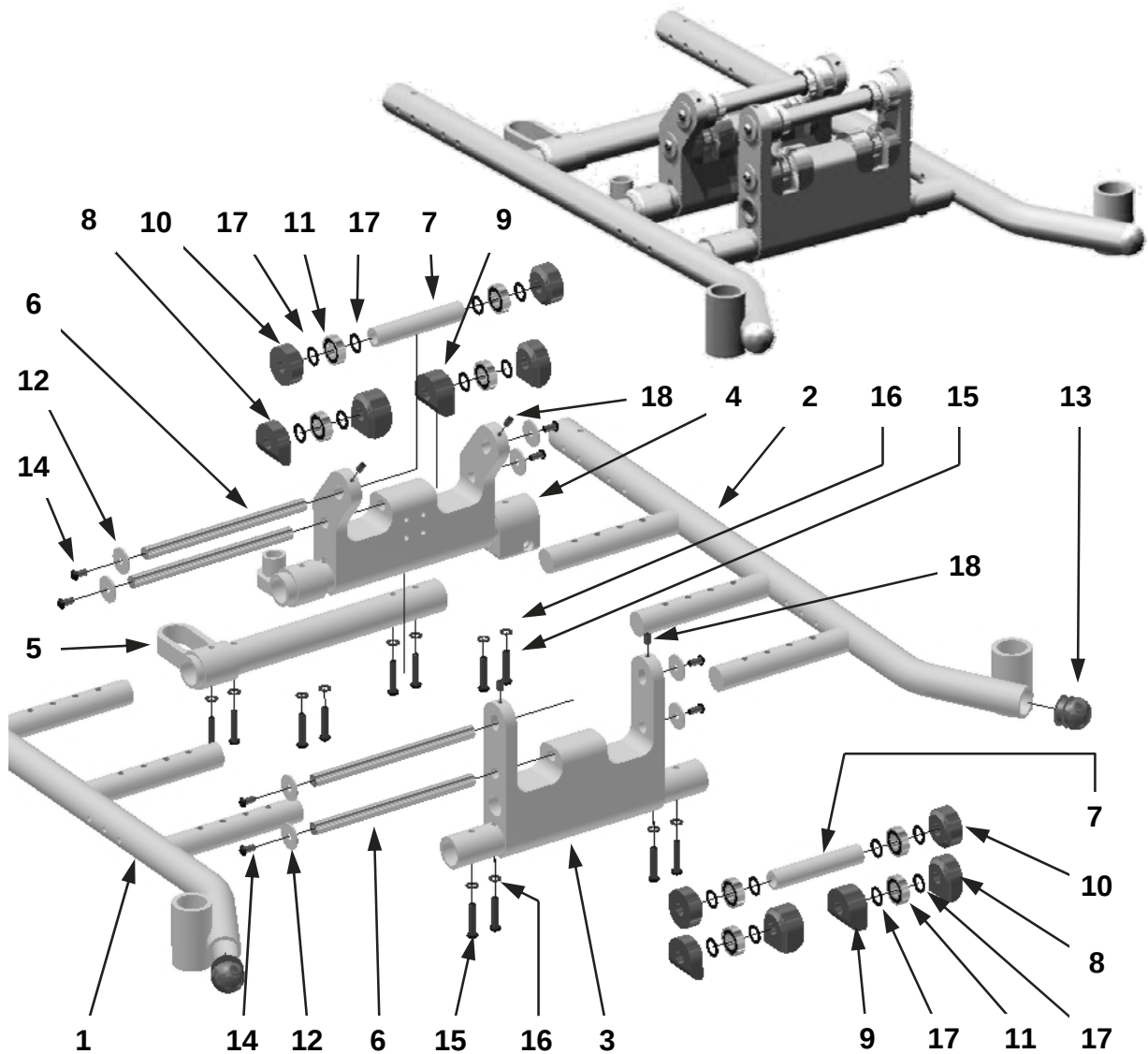

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	17030-RS	Side Frame—Right, Short	1 ea.		17033X	Mounting Pin, X-Wide	1 ea.
	17030-RL	Side Frame—Right, Long	1 ea.	7	17034	Bearing Spacer Tube	1 ea.
	17030-RXL	Side Frame—Right, XLong	1 ea.	8	17035A	Plastic Guide—Bottom Outer	1 ea.
2	17030-LS	Side Frame—Left, Short	1 ea.	9	17035B	Plastic Guide—Bottom Side	1 ea.
	17030-LL	Side Frame—Left, Long	1 ea.	10	17035C	Plastic Guide—Top Side	1 ea.
	17030-LXL	Side Frame—Left, XLong	1 ea.	11	2009	Bearing, 1 1/8" x 1/2"	1 ea.
3	17031-F	Base Tilt Bracket—Front	1 ea.	12	17037	1/4" Fender Washer	1 ea.
	17031-FX	Base Tilt Bracket—Front, X-Wide	1 ea.	13	17038	Tube Cap, 1 1/4"	1 ea.
4	17031-R	Base Tilt Bracket—Rear	1 ea.	14	9031	Screw, 1/4-20 X 1/2" Button Head	1 ea.
	17031-RX	Base Tilt Bracket—Rear, X-Wide	1 ea.	15	9034	Screw, 1/4-20 X 1-1/4" Button Head	1 ea.
5	17032	Base Cross Bar—Rear	1 ea.	16	9074	Washer, 1/4" Flat	1 ea.
	17032X	Base Cross Bar—Rear, X-Wide	1 ea.	17	9114	Washer, 1/2" Flat	1 ea.
6	17033	Mounting Pin	1 ea.	18	9038A	Set Screw, 1/4-20 x 3/8"	1 ea.

SEAT / TILT FRAME—Standard & X-Wide

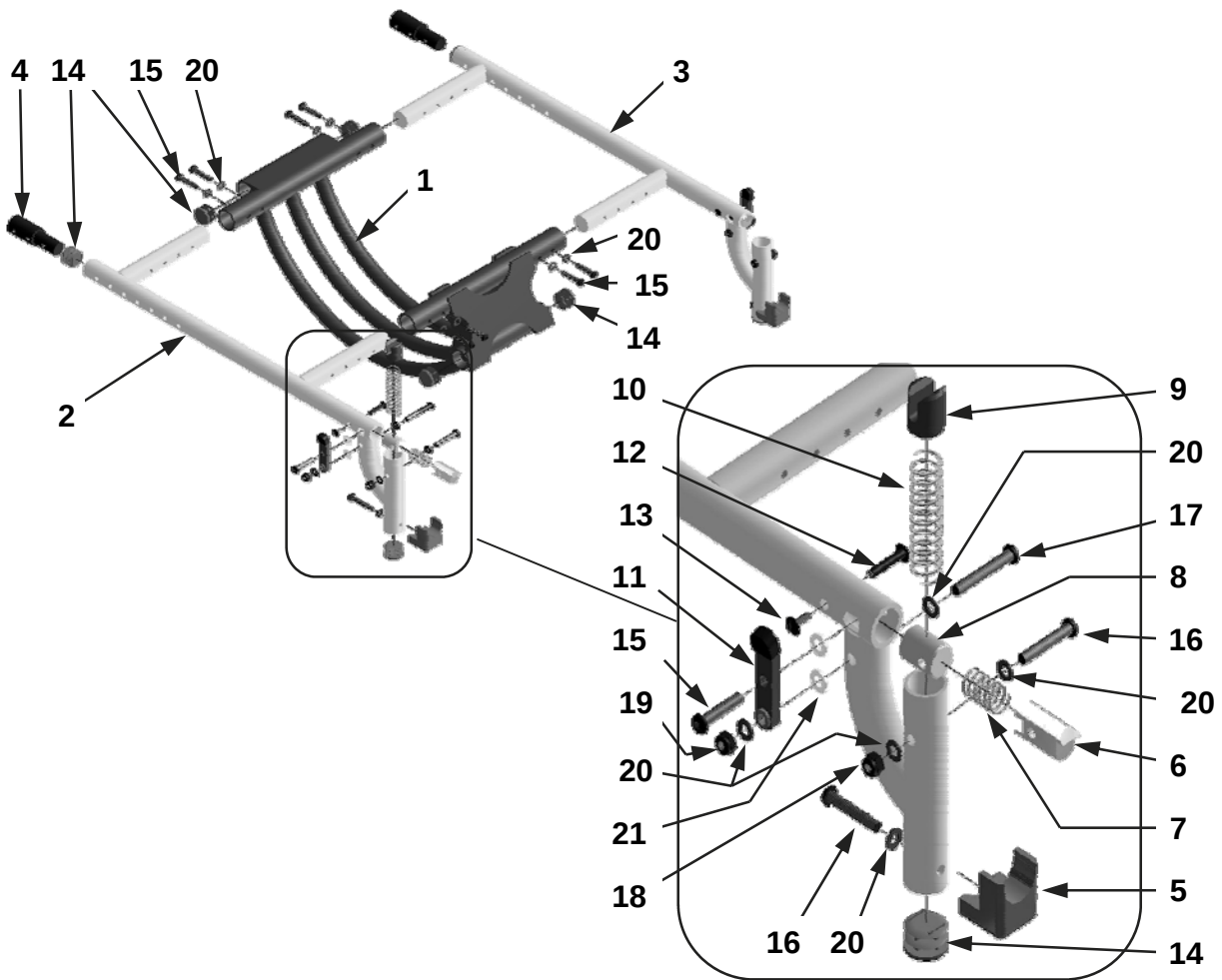

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	17060-S	Tilt Frame Assembly—Short	1 ea.	8	1013	Seat Insert	1 ea.
	17060-L	Tilt Frame Assembly—Long	1 ea.	9	13016	Pop-Up Plug	1 ea.
	17060-X	Tilt Frame Assembly—X-Wide	1 ea.	10	13017	Pop-Up Spring—75 Degree Hanger	1 ea.
2	17061-S	Seat Frame Assembly—Right, Short	1 ea.		13018	Pop-Up Spring—90 Degree Hanger	1 ea.
	17061-M	Seat Frame Assembly—Right, Med.	1 ea.	11	13019	Hanger Release Lever	1 ea.
	17061-L	Seat Frame Assembly—Right, Long	1 ea.	12	9100	Binding Screw—Female	1 ea.
	17061-X	Seat Frame Assembly—Rt, X-Wide	1 ea.	13	9101	Binding Screw—Male	1 ea.
3	17062-S	Seat Frame Assembly—Left, Short	1 ea.	14	9000	Tube Plug, 1"	1 ea.
	17062-M	Seat Frame Assembly—Left, Medium	1 ea.	15	9034	Screw, 1/4-20 X 1-1/4" Button Head	1 ea.
	17062-L	Seat Frame Assembly—Left, Long	1 ea.	16	9035	Screw, 1/4-20 X 1-1/2" Button Head	1 ea.
	17062-X	Seat Frame Assembly—Left, X-Wide	1 ea.	17	9036	Screw, 1/4-20 X 1-3/4" Button Head	1 ea.
4	17070	2" Seat Extension	1 pr.	18	9072	Lock Nut, 1/4-20 Nylon	1 ea.
5	13015	Hanger Support—Pop-Up	1 ea.	19	9073	Lock Nut, 1/4-20 Nylon - Thin	1 ea.
6	1011	Hanger Latch	1 ea.	20	9074	Washer, 1/4" Flat	1 ea.
7	1012	Latch Spring	1 ea.	21	9010	Washer, 1/4" Nylon	1 ea.

GROWTH KITS

Medium Growth Kit Short Frame (10"-14" Wide / 10"-14" Deep) to Medium Frame (12"-16" Wide / 12"-16" Deep)

MEDIUM GROWTH KIT INCLUDES THE FOLLOWING:

- | | | |
|--------------------------------------|------------------------|-------|
| • RIGHT AND LEFT MEDIUM SEAT FRAMES | (#17061-M & 17062-M) | 1 EA. |
| • RIGHT AND LEFT LONG BASE FRAMES | (#17030-RL & 17030-LL) | 1 EA. |
| • LONG TILT FRAME | (#17060-L) | 1 EA. |
| • BACK RIGIDISING CENTER TUBE—MEDIUM | (#17051-M) | 1 EA. |





Long Growth Kit Medium Frame (12"-16" Wide / 12"-16" Deep) to Long Frame (14"-18" Wide / 14"-18" Deep)

LONG GROWTH KIT INCLUDES THE FOLLOWING:

- | | | |
|------------------------------------|----------------------|-------|
| • RIGHT AND LEFT LONG SEAT FRAMES | (#17061-L & 17062-L) | 1 EA. |
| • BACK RIGIDISING CENTER TUBE—WIDE | (#17051-W) | 1 EA. |




GAS SPRING COMPONENTS - X-Wide

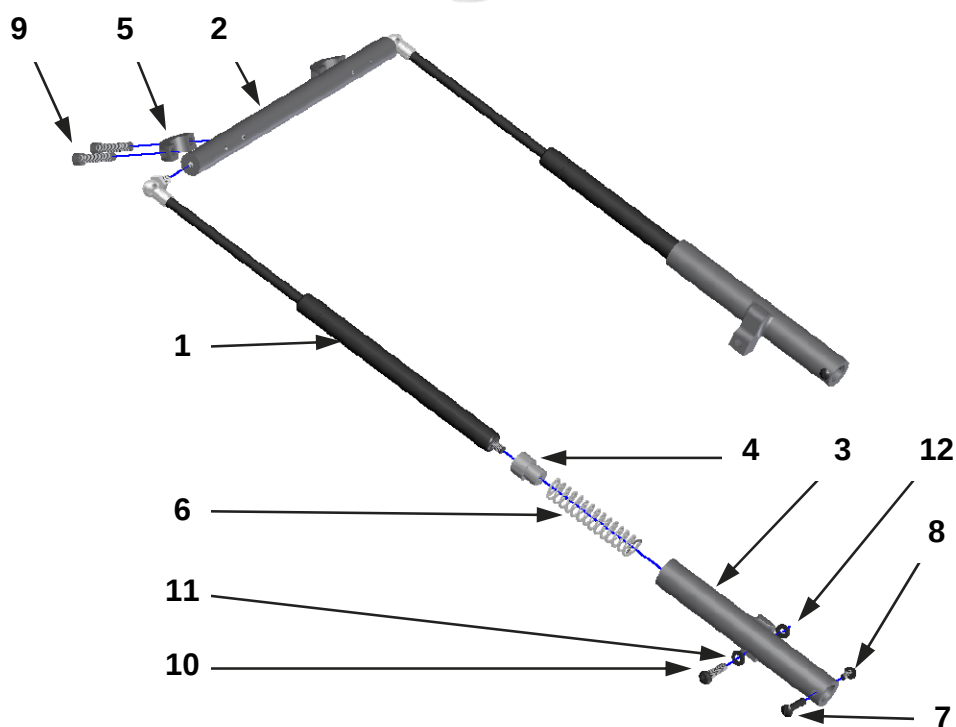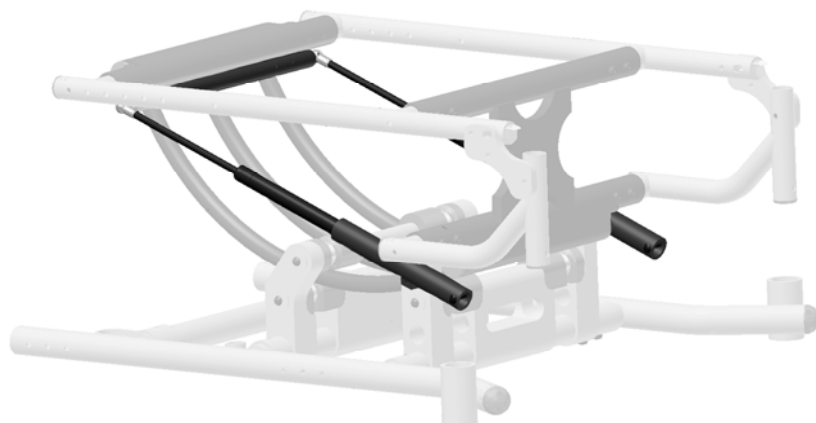


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	17008	Gas Spring	1 ea.	7	9100	Binding Screw—Female	1 ea.
2	17007	Mount Bar—Gas Spring	1 ea.	8	9101	Binding Screw—Male	1 ea.
3	17065	Mount Tube—Gas Spring	1 ea.	9	9062	Screw, 1/4-20 X 1 1/4" Socket Head	1 ea.
4	17066	Spring Seat—Gas Spring	1 ea.	10	9034	Screw, 1/4 x 1 1/4" Button Head	1 ea.
5	17067	Collar Clamp—Spring	1 ea.	11	9074	Washer, 1/4" Flat	1 ea.
6	17068	Spring	1 ea.	12	9010	Washer, 1/4" Plastic	1 ea.

TILT ASSEMBLY—Standard & X-Wide

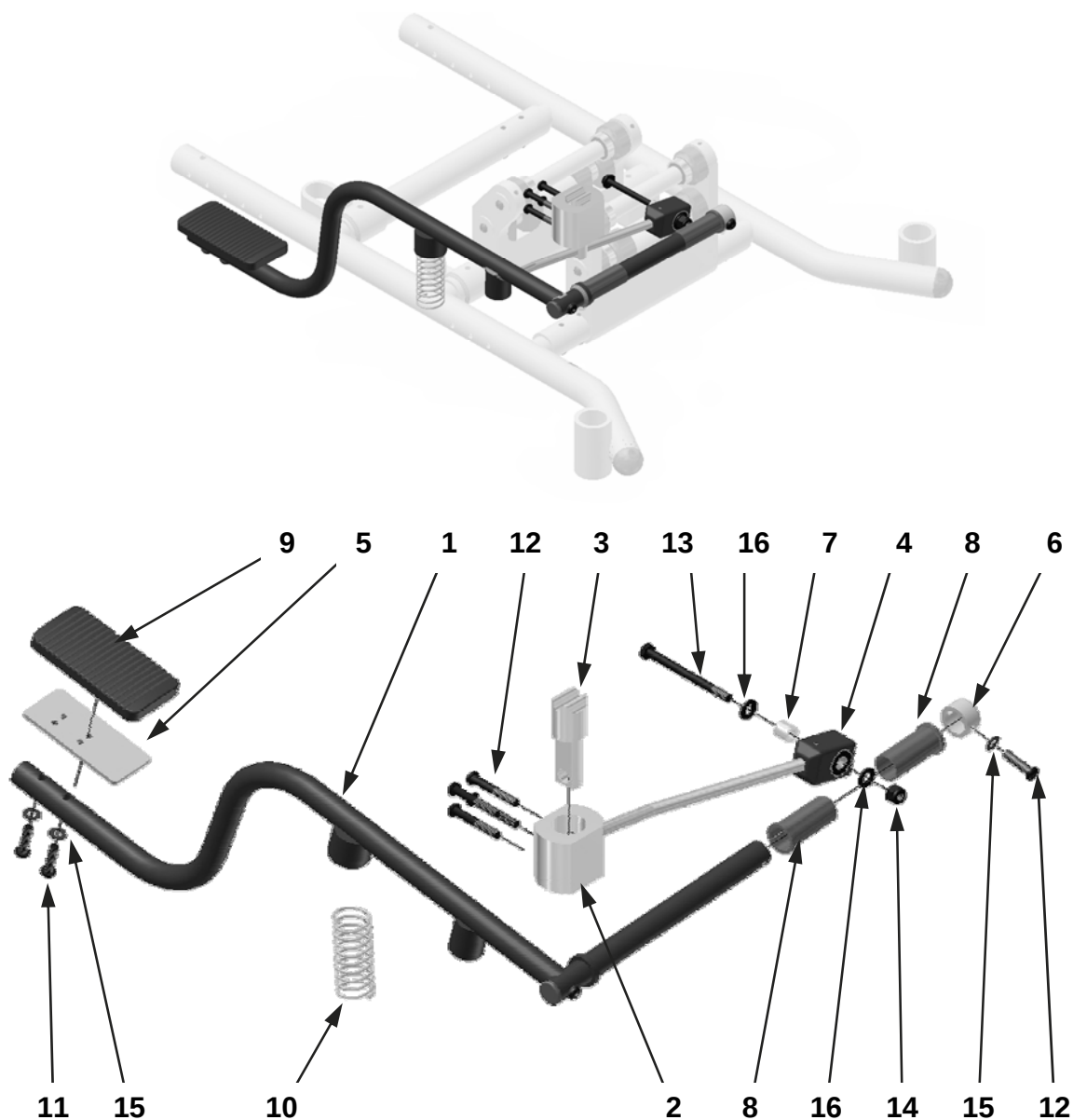

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	17039	Tilt Arm—Standard	1 ea.	8	13070	Nylon Bearing	1 ea.
	17039X	Tilt Arm—X-Wide	1 ea.	9	17048	Pedal Pad	1 ea.
2	17040	Tilt Lock Barrel Housing	1 ea.	10	17049	Tilt Spring	1 ea.
3	17041	Tilt Lock Barrel	1 ea.	11	9033	Screw, 1/4-20 X 1" Button Head	1 ea.
4	17042	Tilt Lock Arm—Standard	1 ea.	12	9036	Screw, 1/4-20 X 1 3/4" Button Head	1 ea.
	17042X	Tilt Lock Arm—X-Wide	1 ea.	13	9084B	Screw, 5/16-18 X 3 1/4" Hex Head	1 ea.
5	17044	Pedal Plate	1 ea.	14	9087	Lock Nut, 5/16-18 Nylon-Thin	1 ea.
6	17045	End Cap—Tilt Arm	1 ea.	15	9074	Washer, 1/4" Flat	1 ea.
7	17046	Spacer Bushing	1 ea.	16	9090	Washer, 5/16 Flat-Thin	1 ea.


HANGERS—Pop Up from serial number CGX-71450

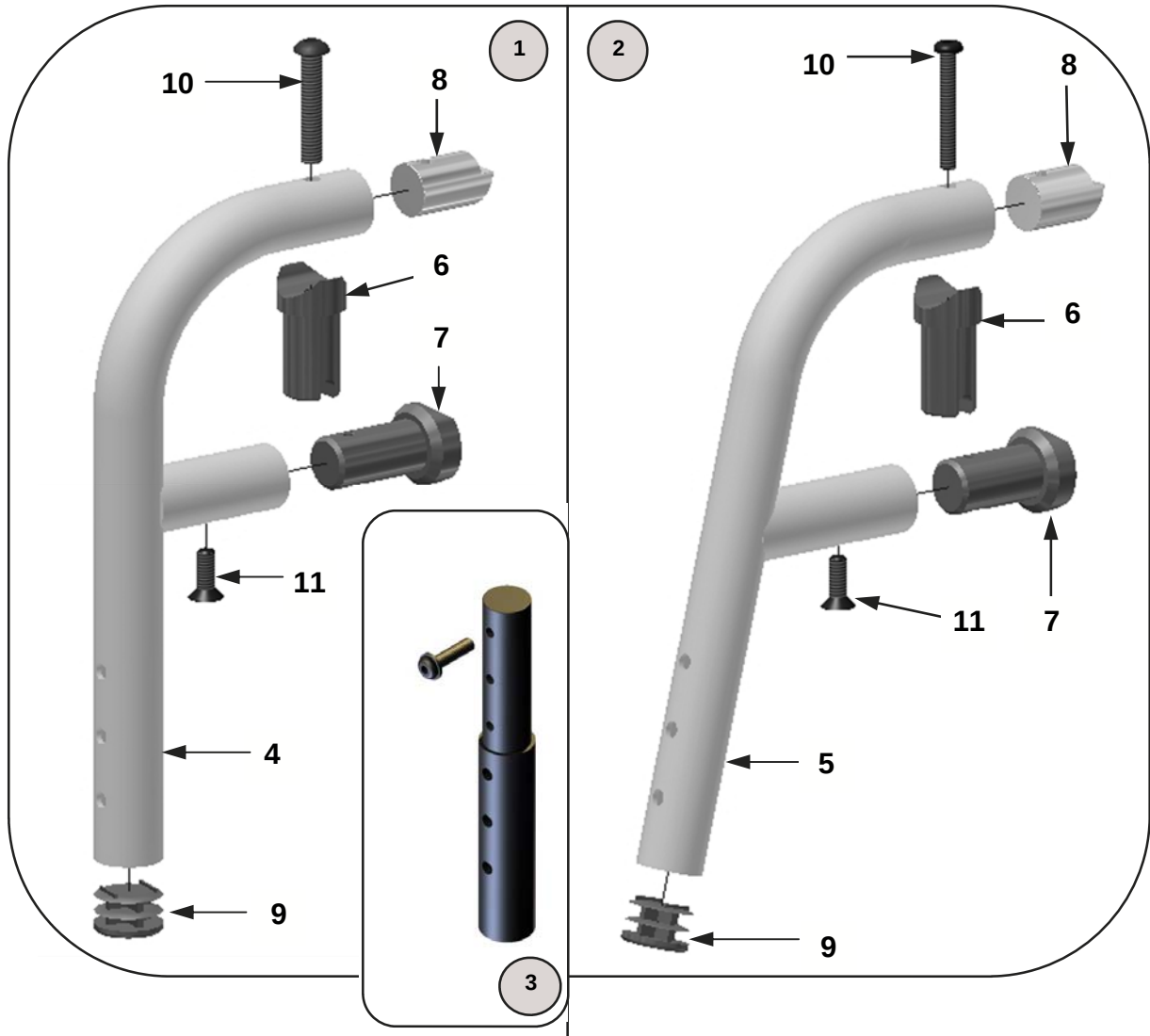

These hangers require footplate assemblies shown on pages 12 & 13.


No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	20086-1	Pop Up Hanger Assembly—90°	1 ea.	4	20089-1	Pop Up Hanger Support Tube—90°	1 ea.
	20086	Pop Up Hanger Assembly—90°	1 pr.	5	20088-1	Pop Up Hanger Support Tube—75°	1 ea.
2	20087-1	Pop Up Hanger Assembly—75°	1 ea.	6	3013	Pivot Fitting—Pop Up	1 ea.
	20087	Pop Up Hanger Assembly—75°	1 pr.	7	1015	Alignment Guide	1 ea.
3	20082-1	Hanger Extension Assembly, 1/2—4 1/2" Long	1 ea.	8	1010	Locking Insert	1 ea.
	20082-2	Hanger Extension Assembly, 1/2—4 1/2" Long	1 pr.	9	9000	Tube Plug, 1"	1 ea.
	20083-1	Hanger Extension Assembly, 2 1/2—6 1/2" Long	1 ea.	10	9036	Screw, 1/4-20 x 1 3/4" Button Head	1 ea.
	20083-2	Hanger Extension Assembly, 2 1/2—6 1/2" Long	1 pr.	11	9046	Screw, 1/4-20 x 3/4" Flat Head	1 ea.

HANGERS—Pop Up Elevating Legrest

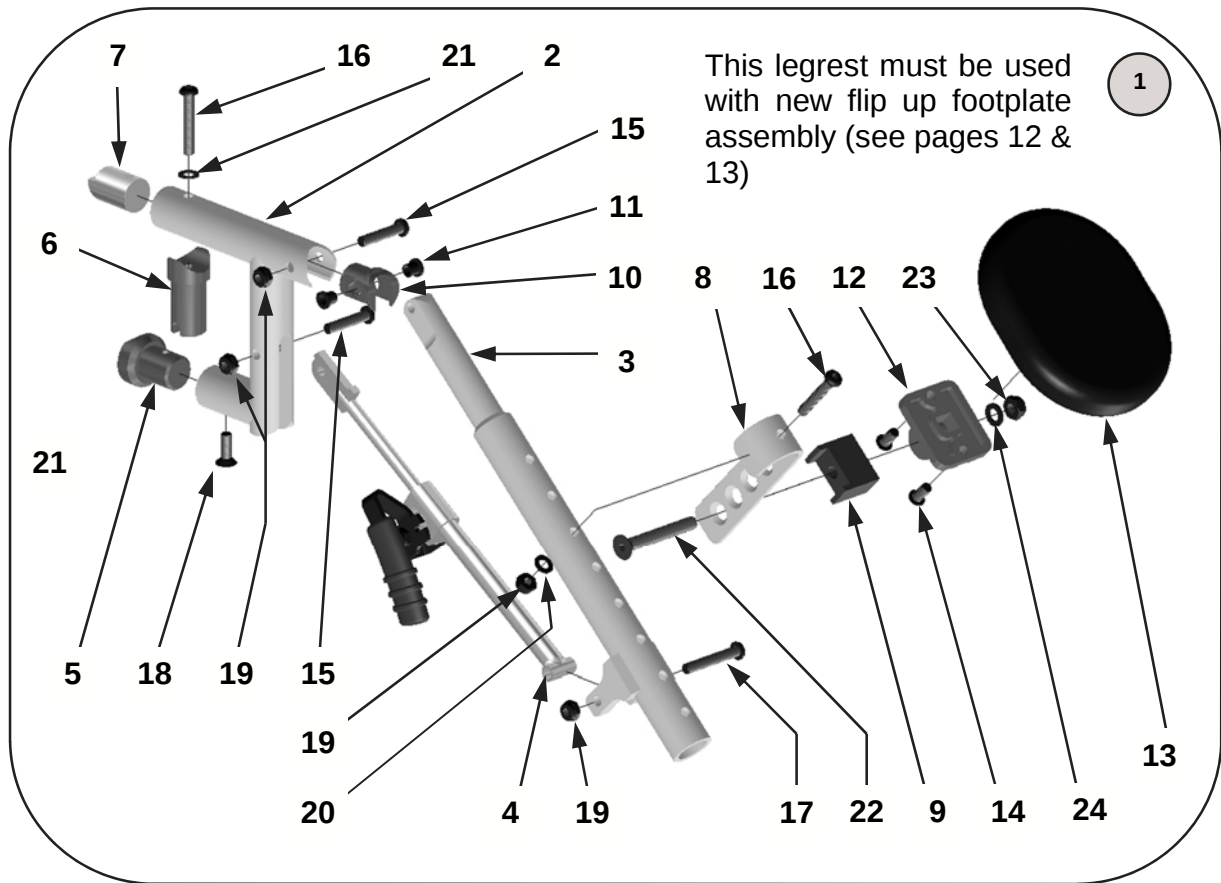

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	14180-L	ELR Assy w/o Pad & Footplate, Left	1 ea.	9	14191	Calf Pad Angle Stop	1 ea.
	14180-R	ELR Assy w/o Pad & Footplate, Right	1 ea.	10	14192	Pivot Insert	1 ea.
	14180	ELR Assy w/o Pads & Footplates	1 pr.	11	14193	Pivot Insert Bushing	1 ea.
	14181-L	ELR Assy w/ Pad , w/o Footplate, Left	1 ea.	12	14209	Calf Pad Mount Plate	1 ea.
	14181-R	ELR Assy w/ Pad , w/o Footplate, Right	1 ea.	13	14210-1	Calf Pad	1 ea.
	14181	ELR Assy w/ Pads, w/o Footplates	1 pr.		14210	Calf Pads	1 pr.
	14182-L	ELR Assy w/ Pad & Footplate, Left	1 ea.	14	9036	Screw, 1/4-20 X 3/4" Button Head	1 ea.
	14182-R	ELR Assy w/ Pad & Footplate, Right	1 ea.	15	9034	Screw, 1/4-20 X 1 1/4" Button Head	1 ea.
	14182	ELR Assy w/ Pads & Footplates	1 pr.	16	9035	Screw, 1/4-20 X 1 1/2" Button Head	1 ea.
2	14211-1	Hanger Weldment, Pop Up	1 ea.	17	9110	Screw, 1/4-20 X 1 5/8" Button Head	1 ea.
	14211-2	Hanger Weldment, Pop Up	1 pr.	18	9046	Screw, 1/4-20 x 3/4" Flat Head	1 ea.
3	14173	Length Adjustment Tube	1 ea.	19	9073	Locknut, 1/4-20 Nylon - Thin	1 ea.
4	14190-R	Locking Mechanism—Right	1 ea.	20	9074	Washer, 1/4" Flat	1 ea.
	14190-L	Locking Mechanism—Left	1 ea.	21	9075	Washer, 1/4" Flat - Thin	1 ea.
5	1015	Alignment Guide	1 ea.	22	9103	Screw, 5/16-18 X 2 1/4" Flat Head	1 ea.
6	3013	Pivot Fitting—Pop Up	1 ea.	23	9088	Locknut, 5/16-18 Nylon - Thin	1 ea.
7	1010	Locking Insert	1 ea.	24	9090	Washer, 5/16 Flat—Thin	1 ea.
8	14194	Pad Mount Hook	1 ea.				

WHEEL LOCKS—Lever Style & Accessories

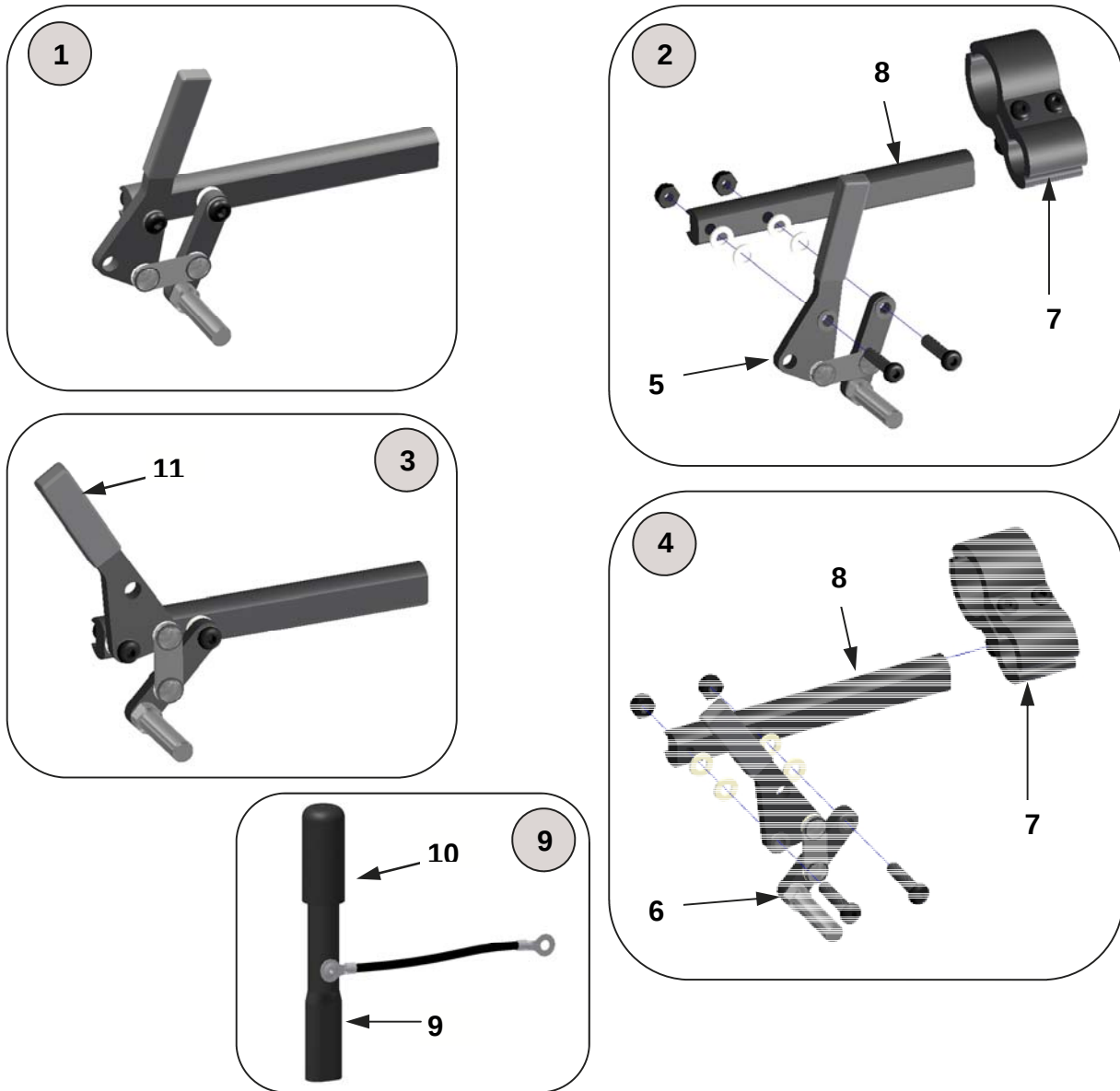

No.	PART #	DESCRIPTION	QTY	No.	PART #	DESCRIPTION	QTY
1	5004-L	Push to Lock Wheel Lock—Left (no clamp)	1 ea.		5024-R	Push to Lock Lever Assembly—Right	1 ea.
	5004-R	Push to Lock Wheel Lock—Right (no clamp)	1 ea.		5024-2	Push to Lock Lever Assemblies	1 pr.
2	17071-L	Push to Lock Wheel Lock / Clamp—Left	1 ea.	6	5025-L	Pull to Lock Lever Assembly—Left	1 ea.
	17071-R	Push to Lock Wheel Lock / Clamp—Right	1 ea.		5025-R	Pull to Lock Lever Assembly—Right	1 ea.
	17071-2	Push to Lock Wheel Locks	1 pr.		5025-2	Pull to Lock Lever Assemblies	1 pr.
3	5007-L	Pull to Lock Wheel Lock—Left (no clamp)	1 ea.	7	17073	Wheel Lock Mounting Clamp	1 ea.
	5007-R	Pull to Lock Wheel Lock—Right (no clamp)	1 ea.	8	5019	Wheel Lock Swivel Arm—Long (6")	1 ea.
4	17072-L	Pull to Lock Wheel Lock / Clamp—Left	1 ea.	9	10064	Lock Extensions	1 pr.
	17072-R	Pull to Lock Wheel Lock / Clamp—Right	1 ea.	10	5016B	Cap for Wheel Lock Extension	1 ea.
	17072-2	Pull to Lock Wheel Locks / Clamps	1 pr.	11	5016C-1	Plastic Cap for Wheel Lock Handle	1 ea.
5	5024-L	Push to Lock Lever Assembly—Left	1 ea.				

WHEELS—Mag Assemblies



Mag Wheel
12", 16" and 18"



Mag Wheel with Tyre & Tube

PART #	DESCRIPTION	QTY	PART #	DESCRIPTION	QTY
11222-1	12" Mag Wheel with Tyre and Tube	1 ea.	11225-1	18" Mag Wheel with Tyre and Tube	1 ea.
11224-1	16" Mag Wheel with Tyre and Tube	1 ea.			

Mag Wheel with Tyre & Insert

PART #	DESCRIPTION	QTY	PART #	DESCRIPTION	QTY
11200-1	12" Mag Wheel with Tyre and Insert	1 ea.	11202-1	18" Mag Wheel with Tyre and Insert	1 ea.
11201-1	16" Mag Wheel with Tyre and Insert	1 ea.			

Wheels Only

Part No.	Description	Qty.
4028	12 1/2" X 2 1/4" Poly Tyre	1 ea.
4029	12 1/2" X 2 1/4" Pneumatic Tyre	1 ea.
4031	16" X 1 3/4" Pneumatic Tyre	1 ea.
4032	18" X 1 3/4" Pneumatic Tyre	1 ea.

Tyres

Part No.	Description	Qty.
4020	12" Mag Rim with Bearings	1 ea.
4021	16" Mag Rim with Bearings	1 ea.
4022	18" Mag Rim with Bearings	1 ea.

Inner Tubes

Part No.	Description	Qty.
4036	12 1/2" X 2 1/4" Inner Tube	1 ea.
4037	16" X 1 3/4" " Inner Tube	1 ea.
4038	18" X 1 3/4" " Inner Tube	1 ea.

Solid Tyre Inserts

Part No.	Description	Qty.
4042	12 1/2" X 2 1/4" Solid Insert	1 ea.
4043	16" X 1 3/4" " Solid Insert	1 ea.
4044	18" X 1 3/4" " Solid Insert	1 ea.

Additional Components

PART #	DESCRIPTION	QTY	PART #	DESCRIPTION	QTY
2009	Bearings, 1 1/8" X 1/2"	1 ea.	4000	Hub Cap for Mag Wheel	1 ea.



Invacare UK Ltd.
Pencoed Technology Park
Pencoed
Bridgend
CF35 5HZ
Tel: 01656 776222
Fax: 01656 776220
E Mail: ordersuk@invacare.com

Invacare Ireland
Unit 5, Seatown Business Campus
Seatown Road
Swords
County Dublin
Ireland
Tel: + 353 1 8107084
Fax: + 353 1 8107085
E Mail: ireland@invacare.com
Web: www.invacare.ie

