

Spare Parts Catalogue

Version: 2019/07/03a NO



Invacare®
Comet^{PRO} / Comet^{ALPINE+} / Comet^{ULTRA}



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

(Information & User Manuals)

(User Manuals Comet Pro / Alpine+ / Ultra)	5
(Label HMI)	6

(Chassis, Shrouds & Lights)

(Front Frame)	7
(Front Shroud)	8
(Rear Frame / Motor Swing)	9
(Rear Shrouds)	10
(Colour Panels (6 pieces))	11
(Light System)	12

(Seating)

(Seats & Headrests)	13
(Seat Post)	14
(Seat Bracket)	15
(Slide Assembly)	16
(Armrests)	17

(Wheels)

(Wheels - 5" rim)	18
(Wheels - 8" rim)	19
(Anti-tipper wheel)	20

(Propulsion)

(Motors & Differentials)	21
--------------------------------	----

(Brakes)

(Brake assembly)	22
------------------------	----

(Steering)

(Steering)	23
(Suspension)	24
(Steering linkage)	25

(Batteries, Cables & Charger)

(Battery & Cables)	26
(Chargers)	27

(Electronic)

(Electronic components)	28
(Powerbox & Limit trigger speed limit)	29

(Options & Accessories)

(Safety)	30
(Warning Triangle)	31



Yes, you can.®

TABLE OF CONTENTS

(Seating)	32
(Walking support)	33
(Rollator bracket)	34
(Storage)	35
(Storage box rear bracket)	36
(Bumpers)	37
(Other Accessories)	38


(User Manuals Comet Pro / Alpine+ / Ultra)

BG	Български	HE	עברית	PL	Polski
CS	Čeština	HR	Hrvatski	PT	Português
DA	Dansk	HU	Magyar	RU	Русский
DE	Deutsch	IT	Italiano	RO	Română
EN	English	JA	日本語	SA	العربية
ES	Español	LV	Latviešu	SK	Slovenčina
ET	Eesti	LT	Lietuvių	SL	Slovenščina
FI	Suomi	NL	Nederlands	SV	Svenska
FR	Français	NO	Norsk	UK	Українська

languages.iso

EN	SP1603455	(User Manual Comet Pro / Alpine+ / Ultra - English)	1
DE	SP1608099	(User Manual Comet Pro / Alpine+ / Ultra - German)	1
IT	SP1608100	(User Manual Comet Pro / Alpine+ / Ultra - Italian)	1
NL	SP1608101	(User Manual Comet Pro / Alpine+ / Ultra - Dutch)	1
NO	SP1608102	BRUKSANVISNING COMET+ COMET ULTRA MANUAL NORSK	1
HR	SP1638425	(User Manual Comet Pro / Alpine+ / Ultra - Croatian)	1
CS	SP1638456	(User Manual Comet Pro / Alpine+ / Ultra - Czech)	1
DA	SP1638457	(User Manual Comet Pro / Alpine+ / Ultra - Danish)	1
ET	SP1638458	(User Manual Comet Pro / Alpine+ / Ultra - Estonian)	1
FI	SP1638459	(User Manual Comet Pro / Alpine+ / Ultra - Finnish)	1
FR	SP1638460	(User Manual Comet Pro / Alpine+ / Ultra - French)	1
HU	SP1638461	(User Manual Comet Pro / Alpine+ / Ultra - Hungarian)	1
LV	SP1638469	(User Manual Comet Pro / Alpine+ / Ultra - Latvian)	1
LT	SP1638505	(User Manual Comet Pro / Alpine+ / Ultra - Lithuanian)	1
PL	SP1638506	(User Manual Comet Pro / Alpine+ / Ultra - Polish)	1
PT	SP1638550	(User Manual Comet Pro / Alpine+ / Ultra - Portuguese)	1
SK	SP1638551	(User Manual Comet Pro / Alpine+ / Ultra - Slovenian)	1
ES	SP1638552	(User Manual Comet Pro / Alpine+ / Ultra - Spanish)	1
SV	SP1638553	(User Manual Comet Pro / Alpine+ / Ultra - Swedish)	1
UK	SP1638554	(User Manual Comet Pro / Alpine+ / Ultra - Ukrainian)	1

(Label HMI)

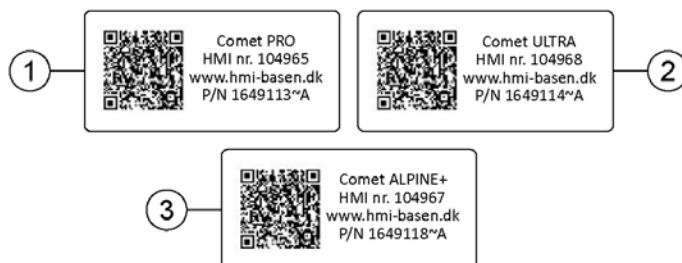


1650303.iso

1	SP1649113	(HMI Label Comet PRO (Sticker Sheet 27 pcs))	1
2	SP1649114	(HMI Label Comet ULTRA (Sticker Sheet 27 pcs))	1
3	SP1649118	(HMI Label Comet ALPINE+ (Sticker Sheet 27 pcs))	1

(Front Frame)


163/038-00

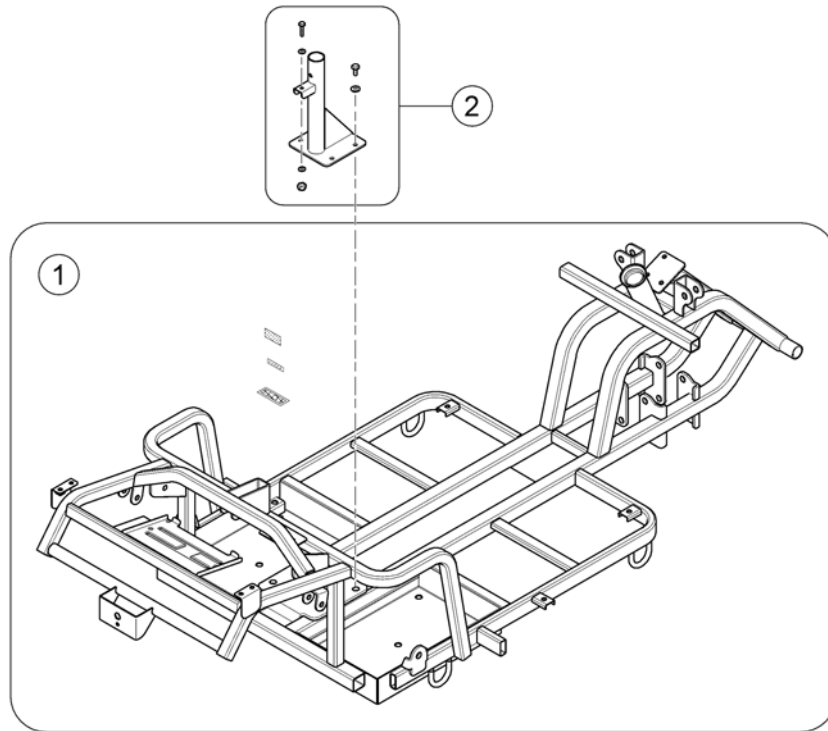

1	SP1636815	(Front frame)	1
2	SP1636817	(Lower seat bracket)	1


(Front Shroud)

1637108-00

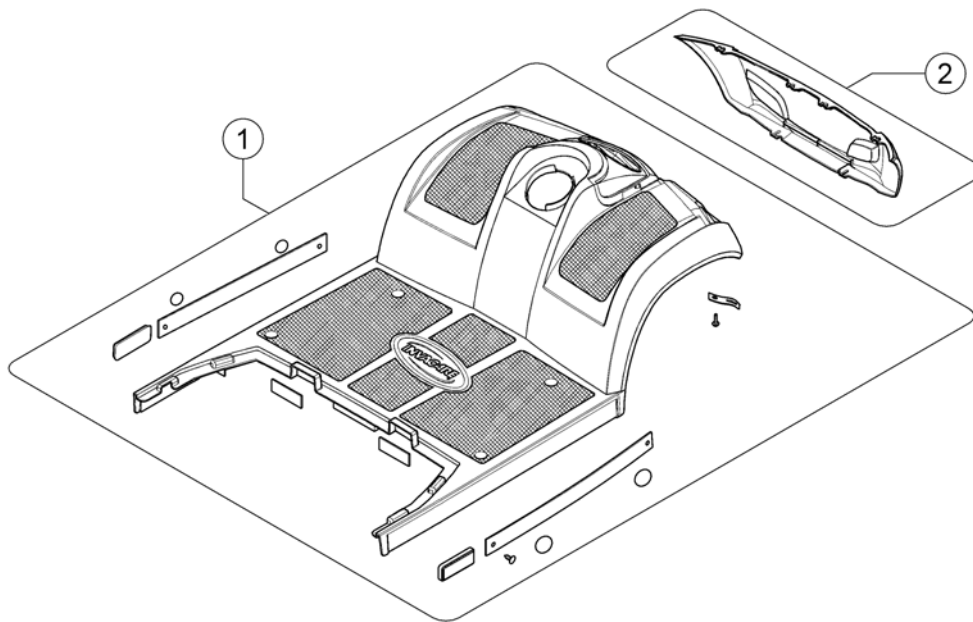

1	SP1636818	FRONTDEKSEL COMET ALPINE+	1
2	SP1608232	DEKSEL STØTFANGER COMET ALPINE+	1

(Rear Frame / Motor Swing)

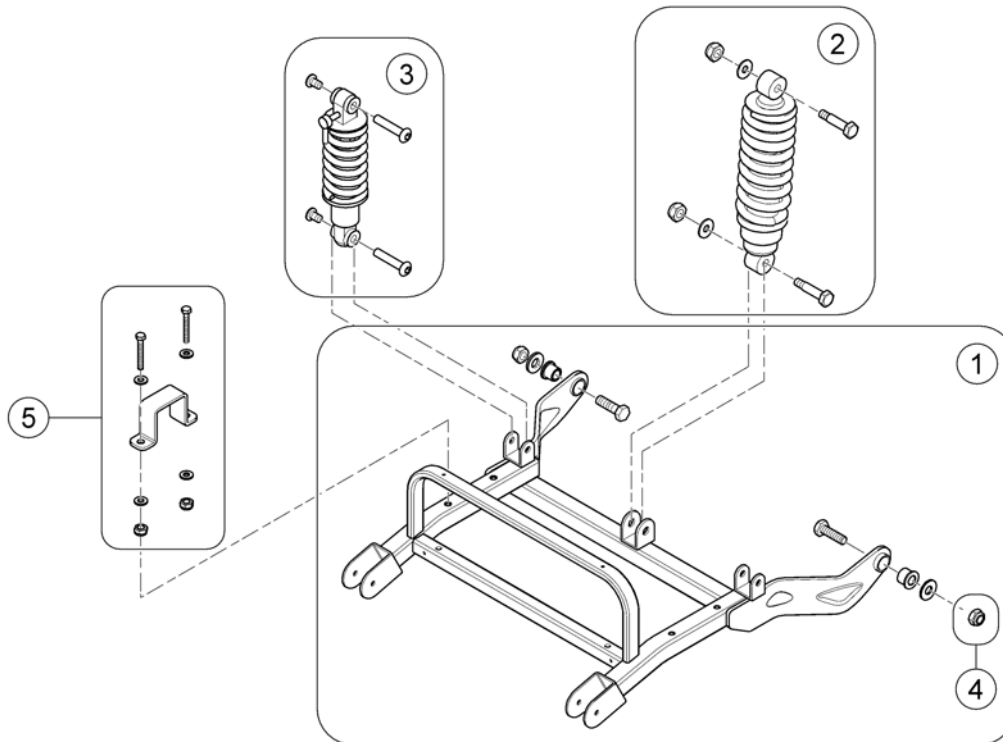


1637105.iso


1	SP1636825	(Rear swing arm cpl.) <i>Design change - fully retrofittable</i>	1
2	SP1645145	(Shock absorber (central), cpl.) <i>Comet PRO / ALPINE+</i>	1
3	SP1636838	(Shock absorber (sideways), cpl.) <i>Comet ULTRA</i>	2
4	SP1505589	(Nylon Nut)	2
5	SP1636837	(Holder, Transaxle, cpl.)	2

(Rear Shrouds)

1637107.000

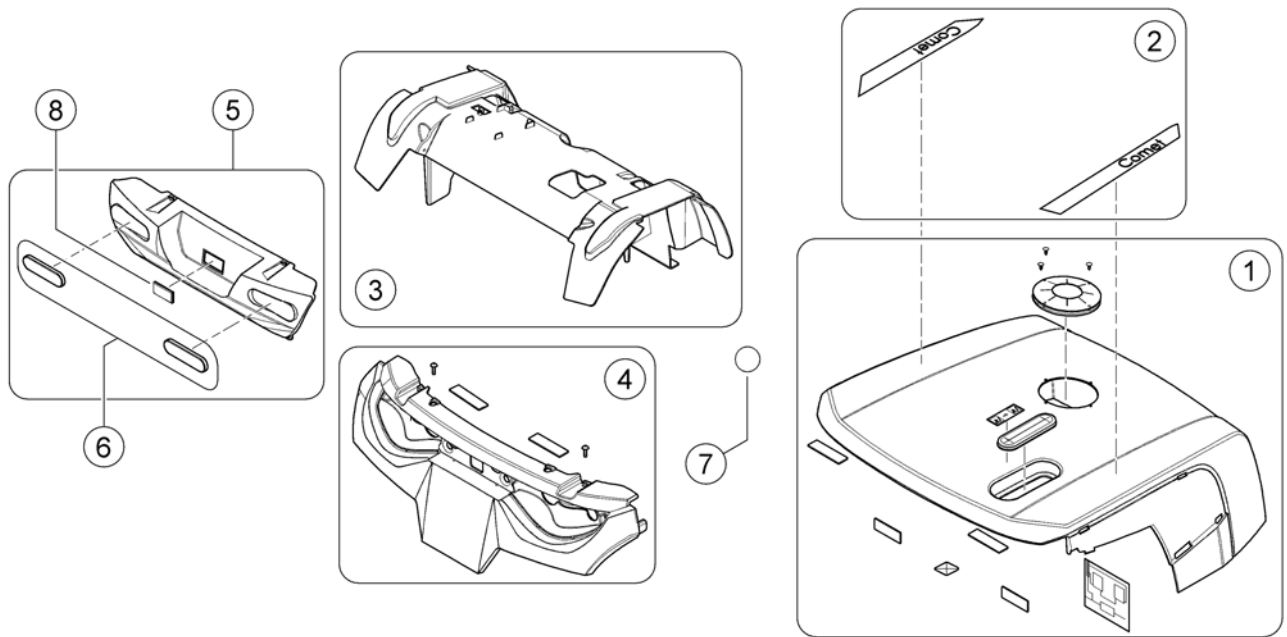

1	SP1608258	DEKSEL OVER BATTERI COMET ALPINE+	1
2	SP1608259	(Label Set - Pro)	1
2	SP1637306	(Label Set - Alpine+)	1
2	SP1637462	(Label Set - Ultra)	1
3	SP1608261	SKJERM BAK COMET ALPINE+	1
4	SP1636839	DEKSEL BAK V STØTFANGER COMET ALPINE+	1
5	SP1608260	DEKSEL BAK V STØTFANGER NEDRE DEL COMET ALPINE+	1
6	SP1518036	REFLEKS BAK COMET	1
7	SP1608262	(Indication stickers, shock absorber (Ultra only))	2
8	SP1521879	LIST BAKRE DEKSEL LITEN COMET	1

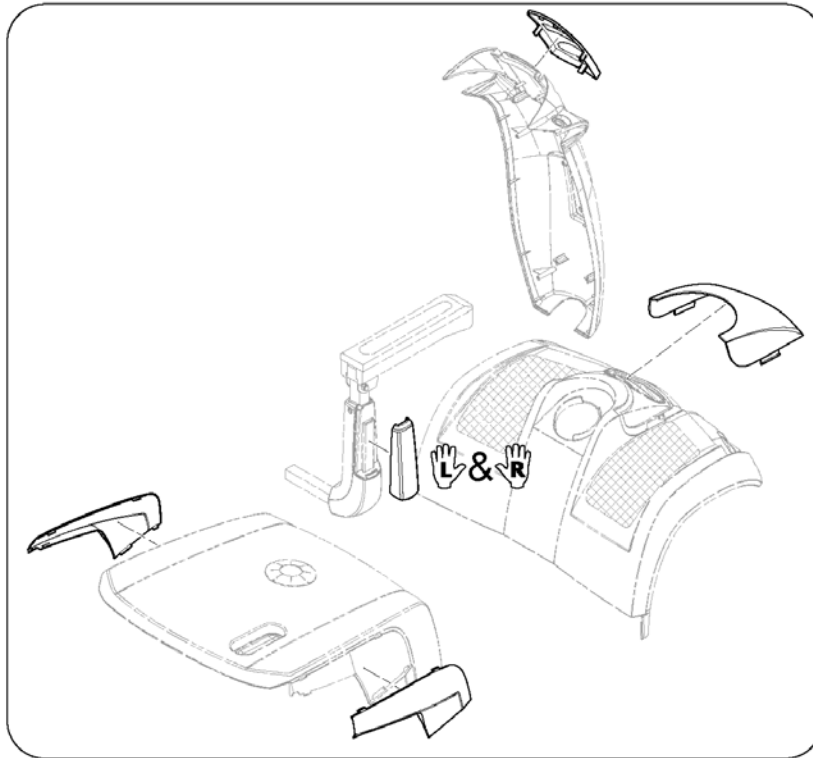
(Colour Panels (6 pieces))

1637123 iso

1	SP1604096	(Color panels - Alium Blue) <i>Replaces SP1608823</i>	1
2	SP1608821	DEKSEL SETT HVIT COMET ULTRA	1
3	SP1608822	DEKSEL SETT GRØNN COMET ULTRA ORION	1
4	-	(Color panels - Sapphire Blue no longer available - please choose Alium Blue.) <i>Replaced by SP1604096</i>	1
5	SP1608831	DEKSEL SETT SØLV ORION PRO	1
6	SP1608832	DEKSEL SETT RØD COMET ULTRA ORION	1
7	SP1637415	DEKSEL SETT CHAMPAGNE COMET ULTRA ORION	1
8	SP1637416	DEKSEL SETT KOBBER FARGE COMET ULTRA ORION	1
9	SP1637417	DEKSEL SETT RED CARMINE COMET ULTRA ORION	1
10	SP1637418	DEKSEL SETT ELECTR BLUE COMET ALPINE+	1
11	SP1637419	DEKSEL SETT CROME FARGE COMET ULTRA ORION	1
12	SP1637420	DEKSEL SETT RØD POLYGON COMET UTRA ORION	1
13	SP1637421	DEKSEL SETT BLÅ POLYGON COMET ULTRA ORION	1
14	SP1637422	DEKSEL SETT HVIT POLYGON COMET UTRA ORION	1



(Light System)

1540472.000

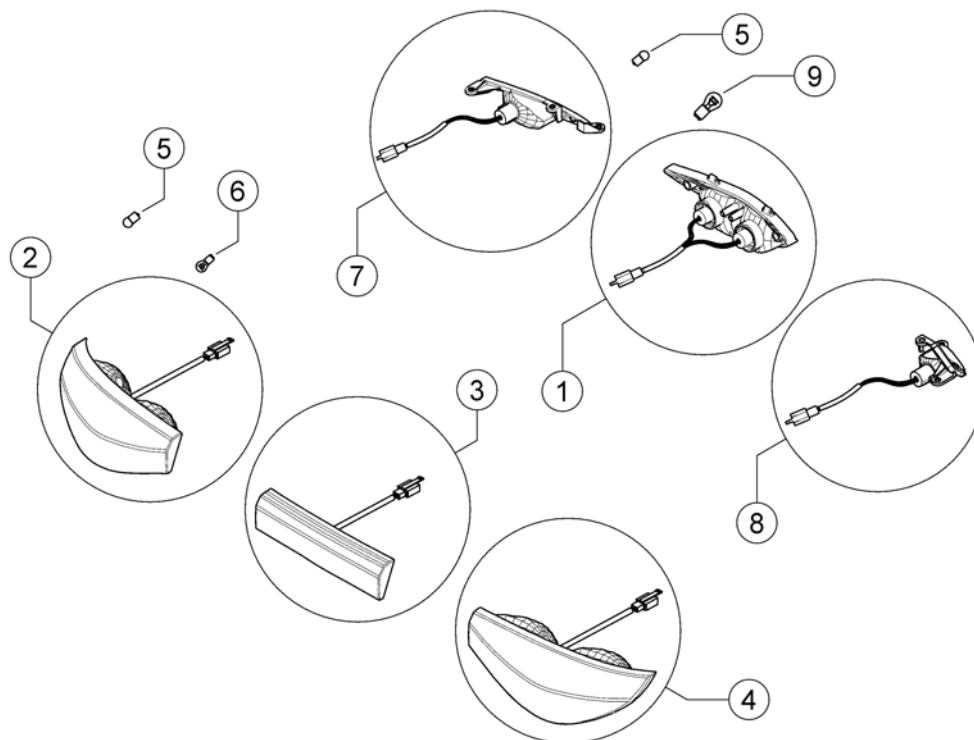

1	SP1527473	HOVEDLYKT KPL COMET	1
2	SP1518034	RETNINGSLYS VE BAK COMET	1
3	SP1518035	BREMSELYS KPL COMET	1
4	SP1518033	RETNINGSLYS HØ BAK COMET	1
5	SP1522030	LYSPÆRE 24V 12W COMET BLINKLYS	4
6	SP1522034	LYSPÆRE 24V 5W COMET BAK	1
7	SP1517733	RETNINGSLYS VE COMET 4W	1
8	SP1517734	RETNINGSLYSHØ COMET 4W	1
9	SP1527474	LYSPÆRE 12V 15W COMET	2


(Seats & Headrests)

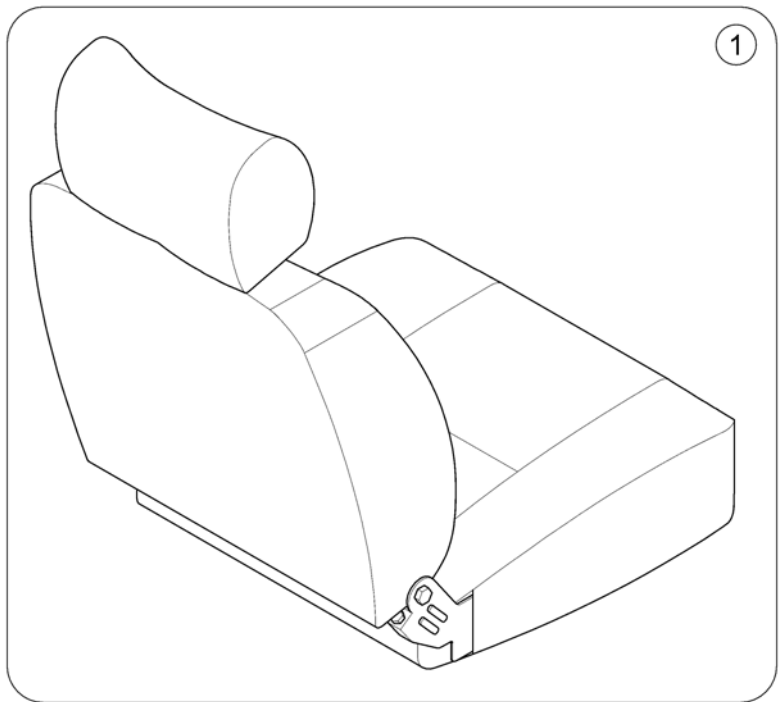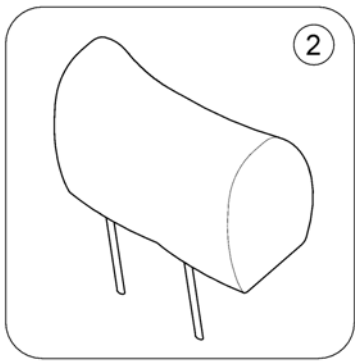

1	SP1608540	SETE COMFORT SB50 ORION PRO 4W COMET	1
1	SP1586116	SETE COMET STD HVIT SØM COMET ALPINE+ <i>Deluxe Seat (DE = Komfort Sitz)</i>	1
1	SP1608556	SETE 20" PREMIUM COMET ALPINE+	1
1	SP1652116	(Seat 20" Deluxe Low (SH 450mm)) <i>Not ULTRA!</i>	1
1	SP1584887	(Seat 20" HD) <i>NORWAY only!</i>	1
1	SP1586115	SETE COMET ULTRA SB66 COMET ULTRA	1
2	SP1529380	HODESTØTTE COMET	1
2	SP1642145	Kopfstütze Premium Sitz	1
2	SP1602195	(Headrest HD Seat)	1


(Seat Post)

167084.00

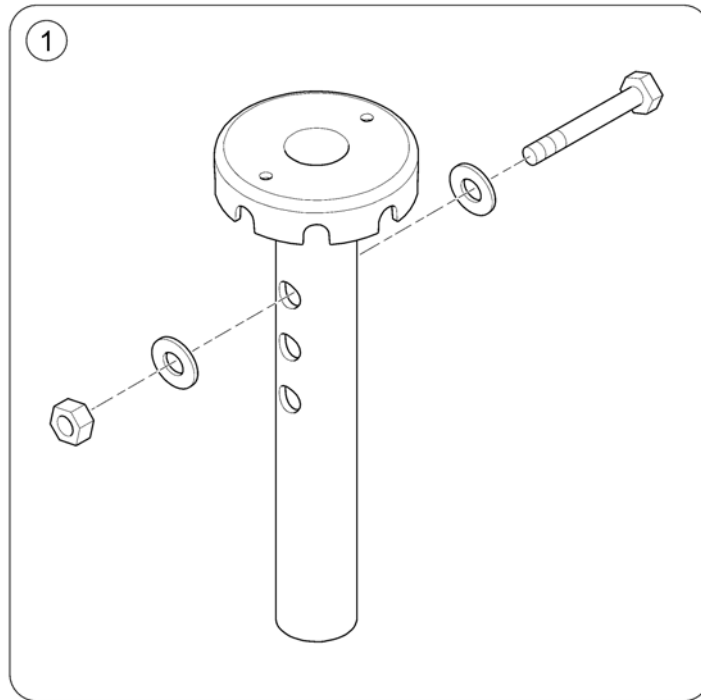

1	SP1521691	SETEBRAKETT M ARMLENSFESTE COMET	1
1	SP1640520	(Seat Post - Comet ULTRA)	1

(Seat Bracket)

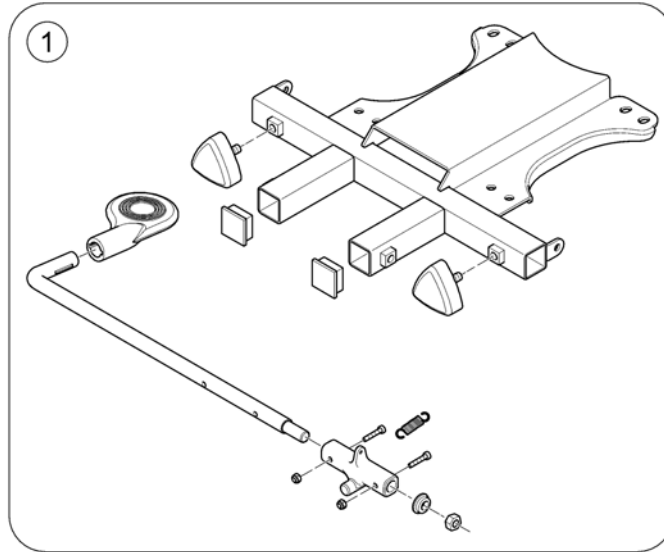


1637085.iso




1	SP1636855	BRAKETT UNDER SETE COMET ALPINE+	1
1	SP1636856	(Seat Bracket Assembly HD, cpl.) <i>Comet ALPINE+ / ULTRA only!</i>	1

(Slide Assembly)

167086-00

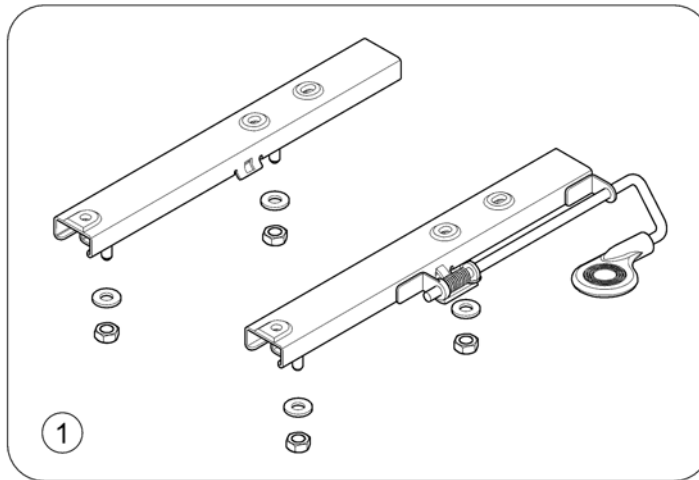

1	SP1521690	GLIDER COMET	1
1	SP1645013	(Slide Assembly - OrionPRO 3W (user weight > 127kg))	1





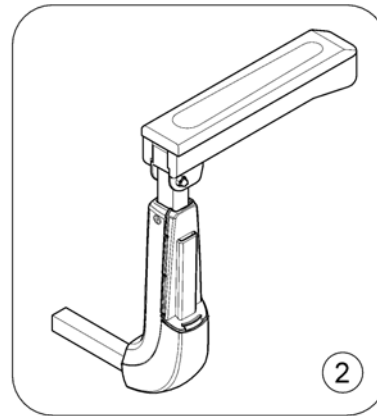
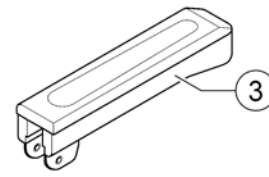
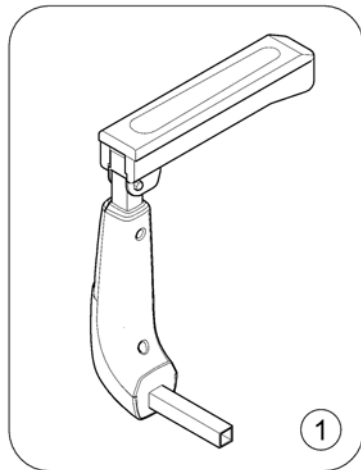


1637396.iso

1	SP1636858	ARMLÉN VENSTRE COMET ALPINE+	1
1	SP1641799	(Armrest HD, cpl. - LH)	1
2	SP1636857	(Armrest, cpl. - RH)	1
2	SP1641800	(Armrest HD, cpl. - RH)	1
3	1509683	(Rest, Handrail)	2

(Wheels - 5" rim)


1639752-00

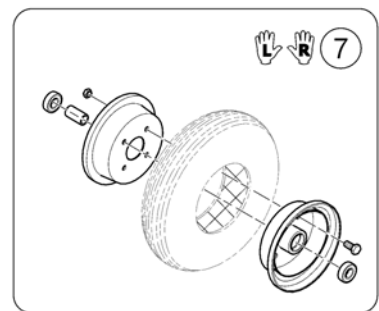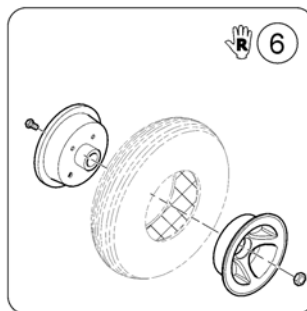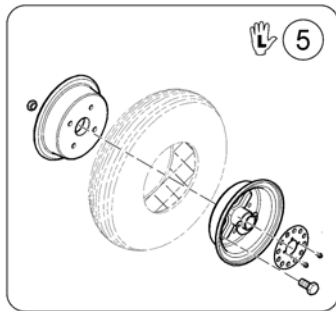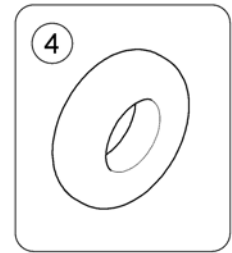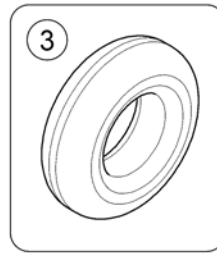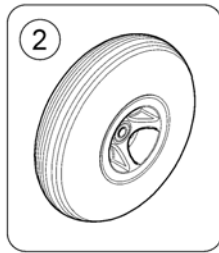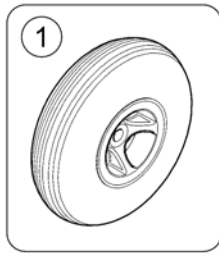

1	SP1608676	BAKHJUL VE KPL LUFT COMET ALPINE+	1
1	SP1637429	BAKHJUL HØ KPL LUFT COMET ALPINE+	1
1	SP1608775	HJUL KOMPAKT BAK VE KPL COMET ALPINE+	1
1	SP1637431	HJUL KOMPAKT BAK HØ KPL COMET ALPINE+	1
2	SP1608594	FORHJUL COMET ALPINE+KPL COMET ALPINE+	1
2	SP1608696	FORHJUL KOMPAKT KPL COMET ALPINE +	1
3	SP1640471	DEKK SORT 4.00-5 STD COMET ALPINE+	1
3	SP1586119	DEKK KOMPACT COMET+ COMET ALPINE+	1
4	SP1527714	SLANGE COMET	1
5	SP1638864	FELG BAK VE COMET ALPINE+ <i>LH</i>	1
6	SP1638838	FELG BAK HØ COMET ALPINE+ <i>RH</i>	1
7	SP1638837	FELG FORAN COMET ALPINE+	1

(Wheels - 8" rim)


1639753.no

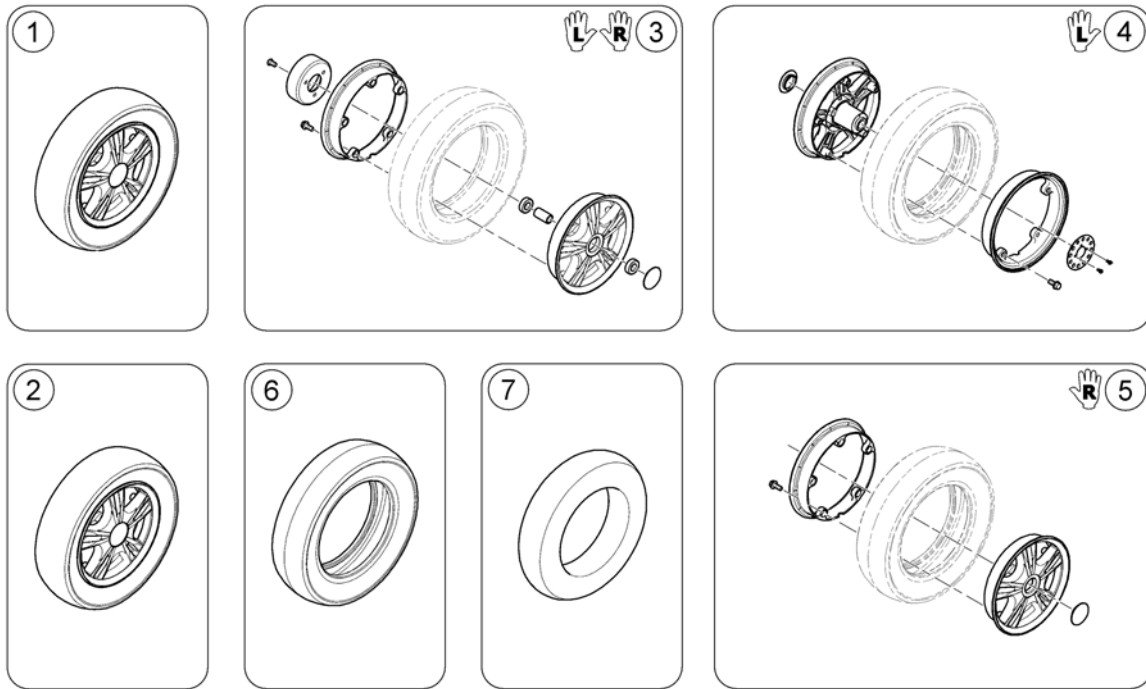

1	SP1608797	FORHJUL LAVPROFIL KPL COMET ALPINE+	1
2	SP1608819	BAKHJUL LAVPROFIL VE KPL COMET ALPINE+	1
2	SP1637433	BAKHJUL LAVPROFIL HØ KPL COMET ALPINE+	1
3	SP1638855	(8" Rim Kit, Front)	1
4	SP1638842	(8" Rim Kit, Rear LH)	1
5	SP1638856	(8" Rim Kit, Rear RH)	1
6	SP1638843	(Tyre 13" for 8" rim)	1
7	SP1638979	(Tube 13" for 8" rim)	1

(Anti-tipper wheel)

167088-00

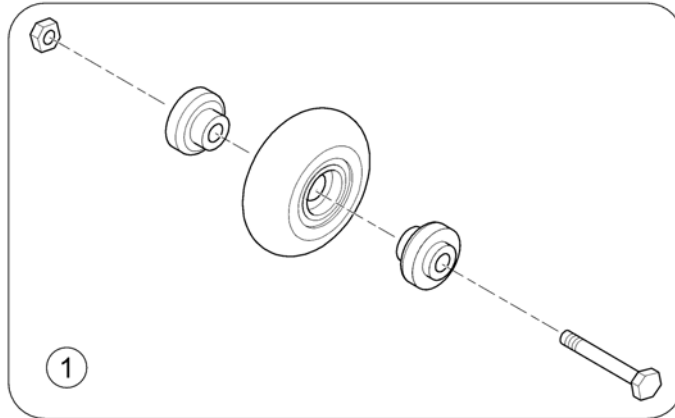


1 SP1522021

HJUL ANTI-TIPPER COMET

1

(Motors & Differentials)


1637108.00

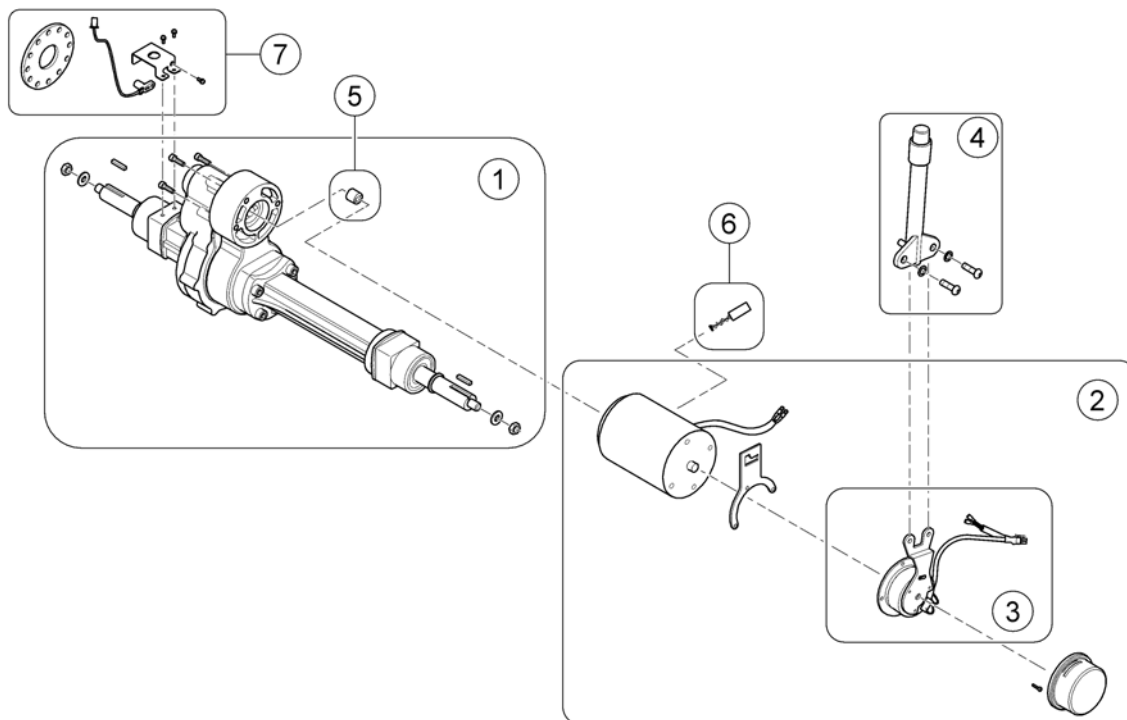

1	SP1636893	(Differential - Pro)	1
1	SP1636894	DIFFERENCIAL COMET+ COMET + COMET ULTRA	1
2	SP1608264	(Motor 10 km/h - Pro)	1
2	SP1608265	(Motor 12.8 km/h - Pro)	1
2	SP1608266	(Motor 15 km/h - Pro)	1
2	SP1636875	MOTOR COMET ALPINE+ULTRA COMET ALPINE+	1
3	SP1636879	(Clutch & Brake) <i>10 km/h (Pro)</i>	1
3	SP1636891	(Clutch & Brake) <i>12.8 & 15 km/h (Pro)</i>	1
3	SP1636892	(Clutch & Brake) <i>10 km/h (Alpine+ / Ultra)</i>	1
4	SP1505672	FRIKOPLINGSHENDEL COMET	1
5	SP1571584	(Guide)	1
6	SP1636876	(Carbon brush) <i>10 km/h (Pro)</i>	1
6	SP1636877	(Carbon brush) <i>12.8 & 15 km/h (Pro)</i>	1
6	SP1636878	(Carbon brush) <i>10 km/h (Alpine+ / Ultra)</i>	1
7	SP1643083	(Kit, Speed Sensor, cpl.) <i>replaces item SP1636874</i>	1

(Brake assembly)


163822-00

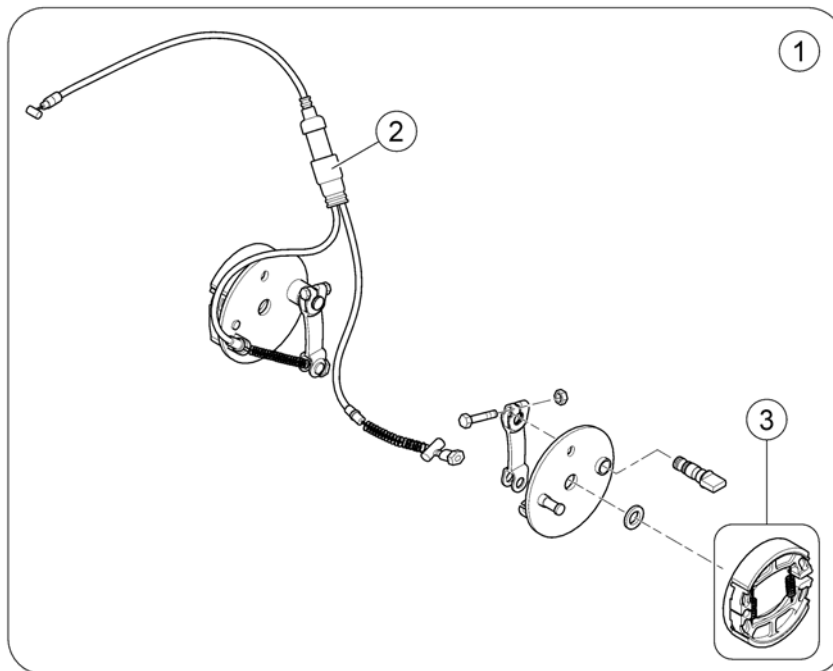

1	SP1636873	(Brake Assembly)	1
2	SP1608231	(Brake Cable)	1
3	SP1518418	(Brake Shoes, cpl. (95mm))	1


(Steering)

1637057/000

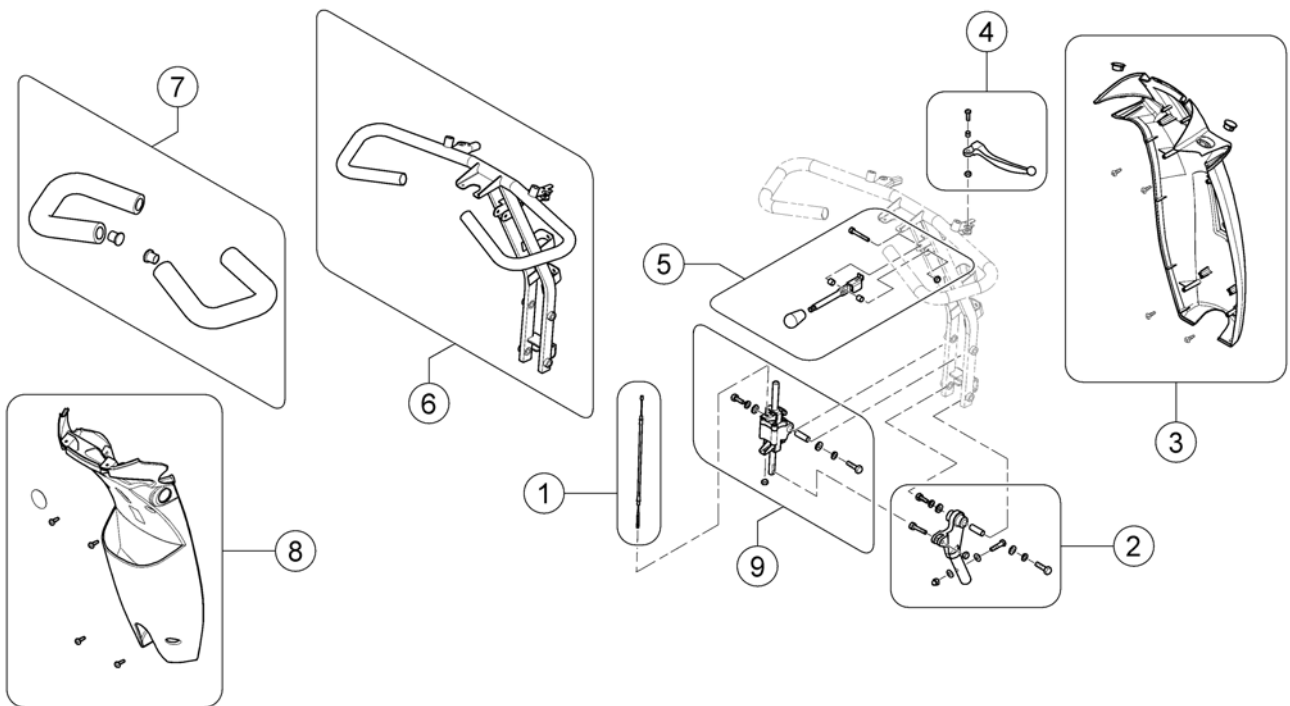

	SP1637540	(Reflective Sticker)	1
1	SP1608201	(Cable, steering adj.)	1
2	SP1636812	(Joint, steering adjustment, cpl.)	1
3	SP1608205	DEKSEL STYRE COMET ALPINE+	1
4	SP1636804	(Lever, Brake - RH)	1
5	SP1636813	(Lever, steering adjustment)	1
6	SP1608184	(Steering)	1
7	SP1636767	(Grip, Handle Bar, cpl.)	1
8	SP1636814	DEKSEL RATTSTAMME BAKRE COMET ALPINE+	1
9	SP1608202	(Steering adjustment Assembly, cpl.)	1


(Suspension)

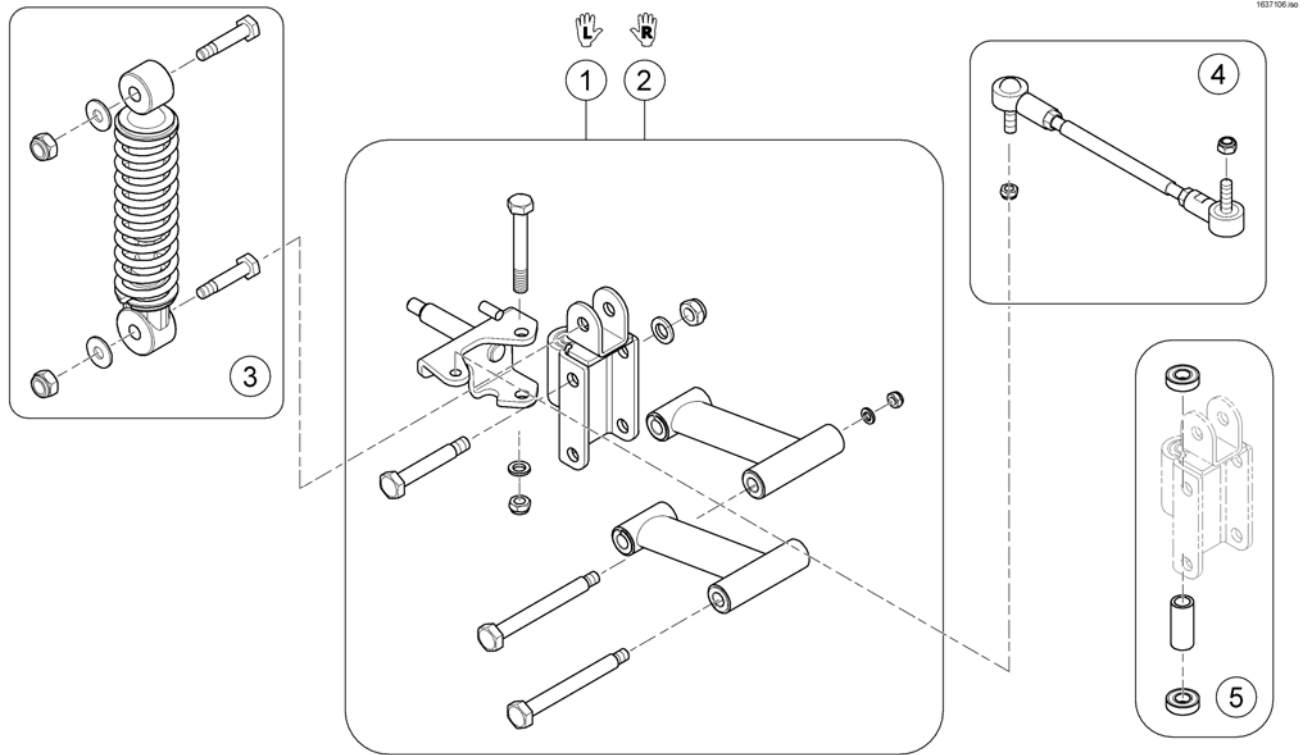

1	SP1636860	(Suspension - LH)	1
2	SP1636862	(Suspension - RH)	1
3	SP1636864	(Front suspension, cpl.)	1
4	SP1636872	(Rod Assembly, cpl.)	1
5	SP1636859	(Bearing Kit)	1

(Steering linkage)


1637083.00

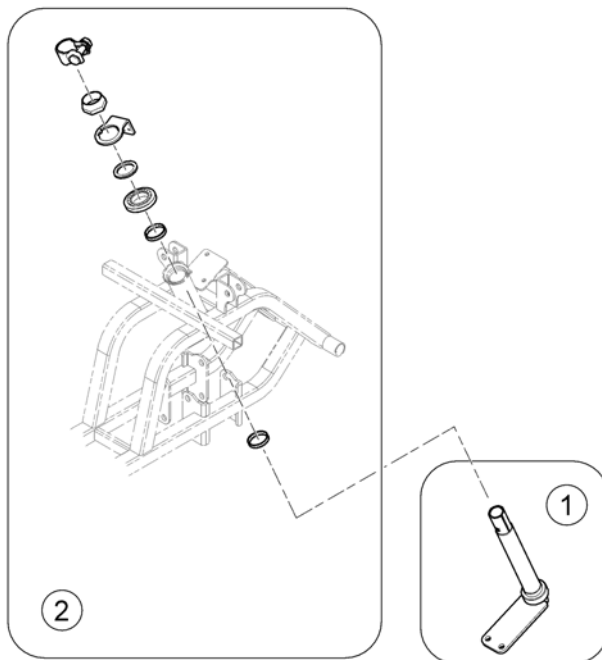

1	SP1608211	(Steering bar assembly)	1
2	SP1636816	(Bearing Kit)	1


(Battery & Cables)

1037121.000

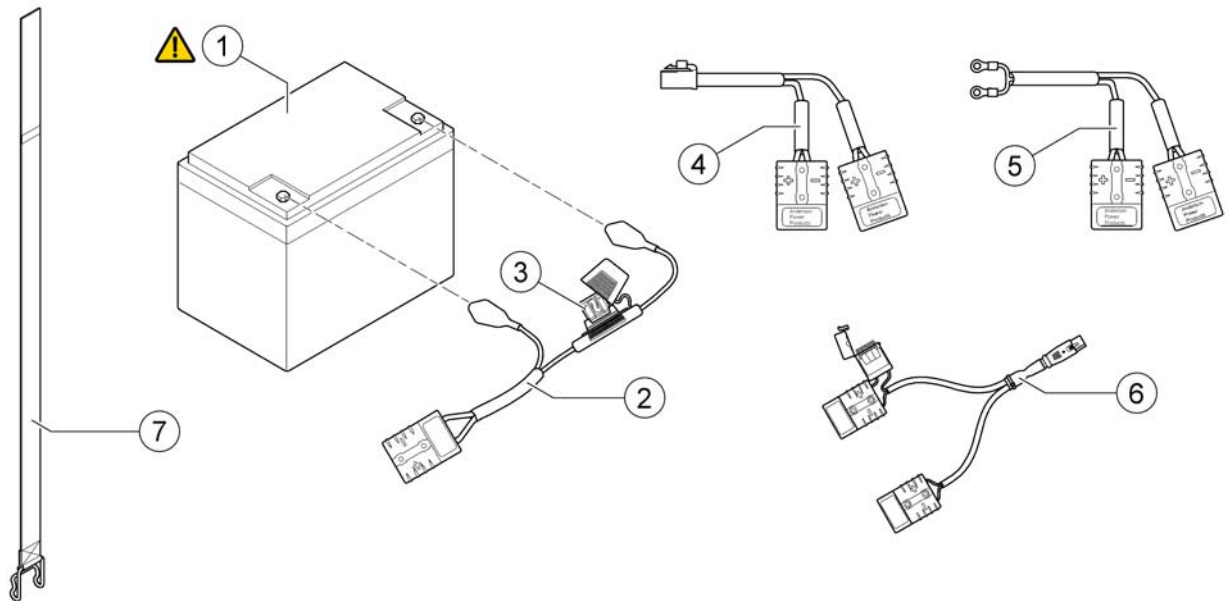

!	-	(Important advice! Replace batteries in pairs. Never mix: manufactures, date codes or technologies (Gel with AGM batteries))	0
1	SP1522026	(Battery, 12V/75Ah C20, AGM)	2
2	SP1636819	(Fuse battery cable (70A)) <i>PRO</i>	1
2	SP1636822	(Fuse battery cable (100A)) <i>ALPINE+ & ULTRA</i>	1
3	SP1636823	(Fuse 70A)	1
3	SP1636824	(Fuse 100A)	1
5	SP1637918	(Wire, Battery Output - Comet ALPINE+ / ULTRA)	1
4	SP1608210	(Wire, Battery Output - Comet PRO)	1
6	SP1641608	(Off board charging cable V0)	1
7	SP1521758	BATTERISTROPP COMET	1

(Chargers)

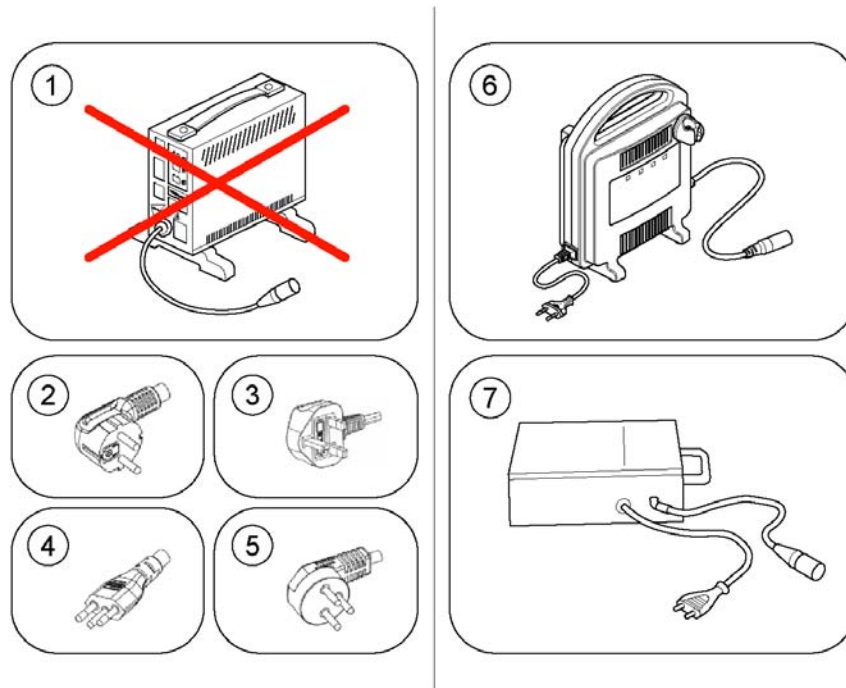


16-40673-180

1	-	(No longer available.)	1
2	P21079	(Power Cord - EU Plug (IEC 60210-C13 / CEE 7/7 Type F 'Schuko'))	1
3	P21080	(Power Cord - UK Plug (IEC 60320-C13 / BS 1363 Type G))	1
4	1443841	(Power Cord - Swiss Plug (IEC 60320-C13 / SEV 1011 Type J))	1
5	SP1553849	(Power Cord - Israel Plug (IEC 60320-C13 / SI 32 Type H))	1
6	SP1606541	(Charger OptiCharge 8A - EU Plug) [1343]	1
6	SP1606543	(Charger OptiCharge 8A - UK Plug) [1343]	1
6	SP1606544	LADER OPTICHARGE 10A OPTICHARGE [1313]	1
6	SP1606545	(Charger OptiCharge 10A - UK-Plug) [1313]	1
7	5813987	(Charger GI Popular 8A (IP54) - EU Plug) [1388]	1

(Electronic components)


1637110.iso

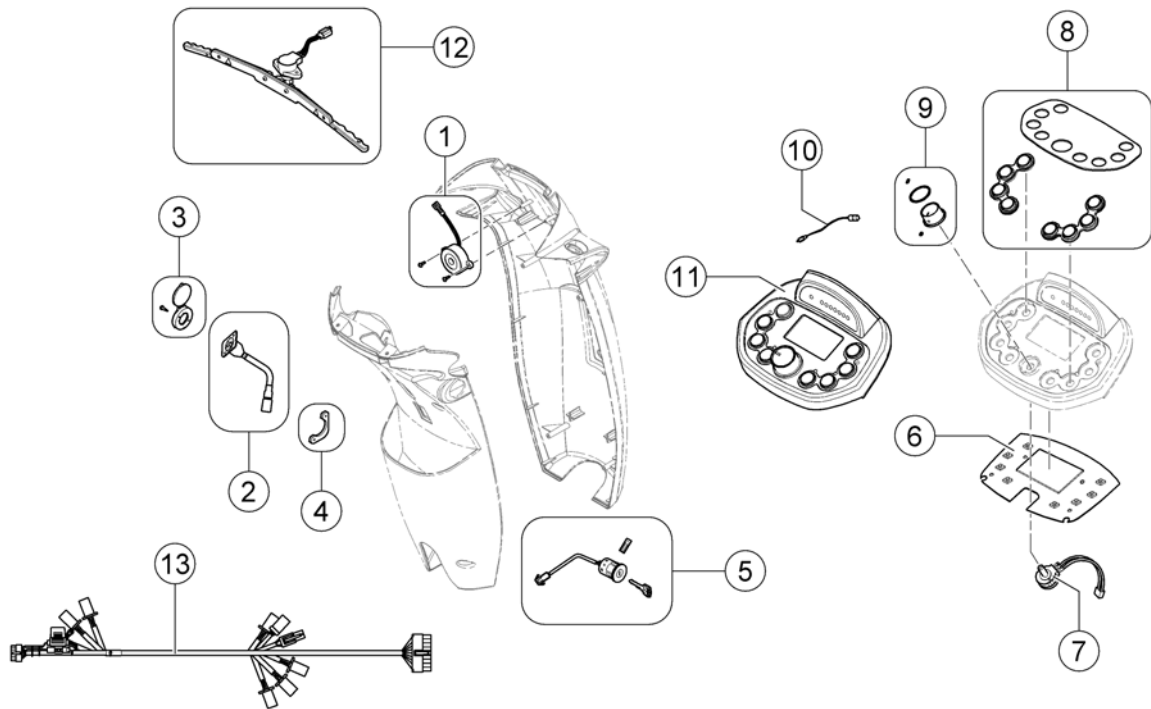

1	SP1506153	BUZZER COMET ALPINE+	1
2	SP1523543	(Charger-Inlett)	1
3	SP1523534	DEKSEL LADEKONTAKT COMET COMET ALPINE +	1
4	SP1523544	PLATE STYRE COMET	1
5	SP1505451	(Key Cylinder (Standard))	1
6	SP1608208	PRINTKORT STYRE COMET ALPINE +	1
7	SP1505462	POTENSIOMETER GASSREG COMET ALPINE+	1
8	SP1637215	(Display cover & rubber keys (LCD Display))	1
9	SP1608207	(Knob, Speed control)	1
10	SP1608204	(Temperature sensor)	1
11	SP1608185	PANEL KONPLETT M LCD COMET ALPINE+	1
12	SP1608200	GASSHENDEL KPL COMET	1
13	SP1608212	(Wire Harness - Pro)	1
13	SP1608229	(Wire harness - Alpine+ & Ultra)	1
-	SP1521577	SIKRING 10A COMET	1

(Invacare® Comet PRO / ALPINE+ / ULTRA) | (Electronic)
(Powerbox & Limit trigger speed limit)


1637122.00

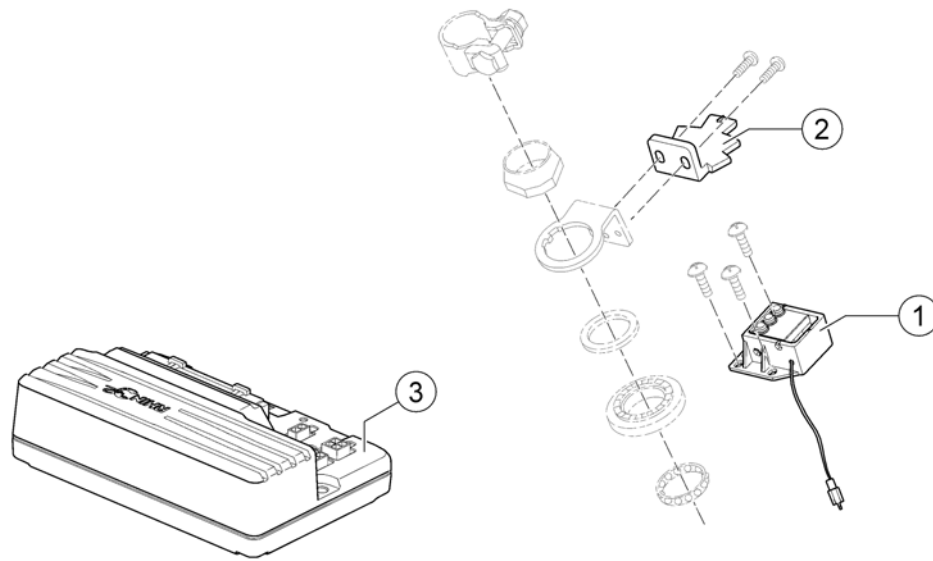

-	-	(For Mounting the Speed Reduction Switch, the Adjustment Gauge SP1581766 is needed!)	1
-	SP1581766	(Adjustment Gauge, Speed Reduction Switch)	1
1	SP1556266	BRYTER FARTSREDUKSJON COMET 2 GENERASJON <i>Comet Pro/Alpine+/Ultra 10 km/h</i>	1
1	SP1556281	(Speed Limit Switch Box) <i>Comet Pro 12.8 & 15 km/h</i>	1
2	SP1608230	(Trigger, limit switch)	1
3	SP1553399	(Controller, Rhino2, 120A, DS120) <i>Comet Pro</i>	1
3	SP1553400	POWERMODUL MODUL COMET ALPINE FRA OKT2011 <i>Comet Alpine+/Ultra</i>	1

(Safety)


1540378 no

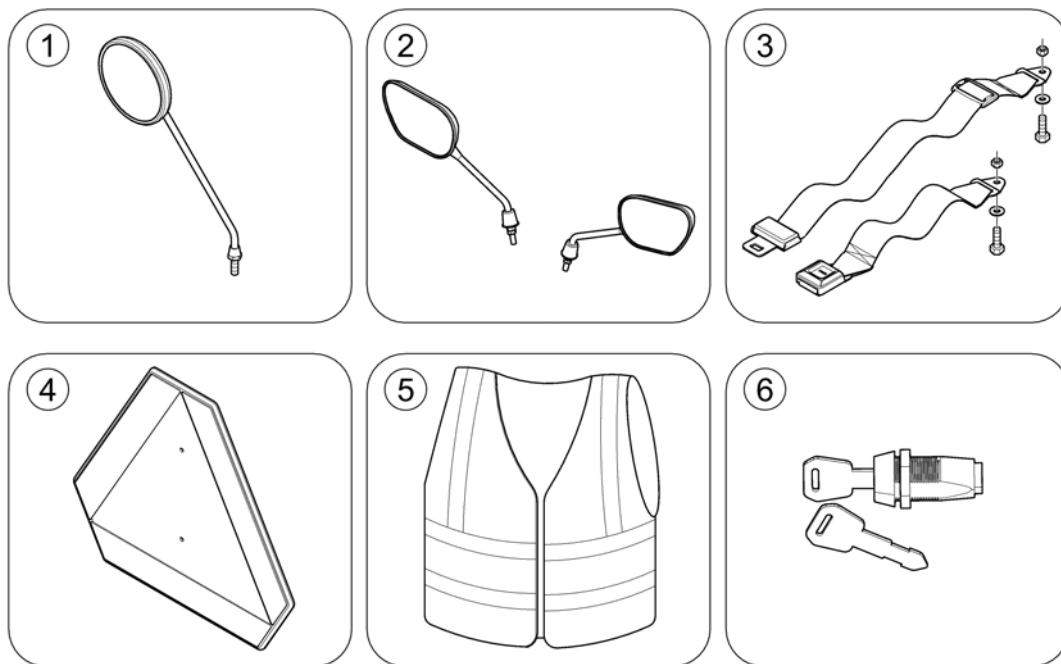

1	SP1518241	SPEIL COMET	1
2	SP1637911	(High-end rear mirror set)	1
3	SP1518274	HOFTEBELTE COMET	1
4	-	(Warning Triangle)	1
5	SP1521592	(Safety vest)	1
6	SP1543154	LÅS M 2 NØKLER UNIK COMET	1

(Warning Triangle)


OHComFig.8.iso

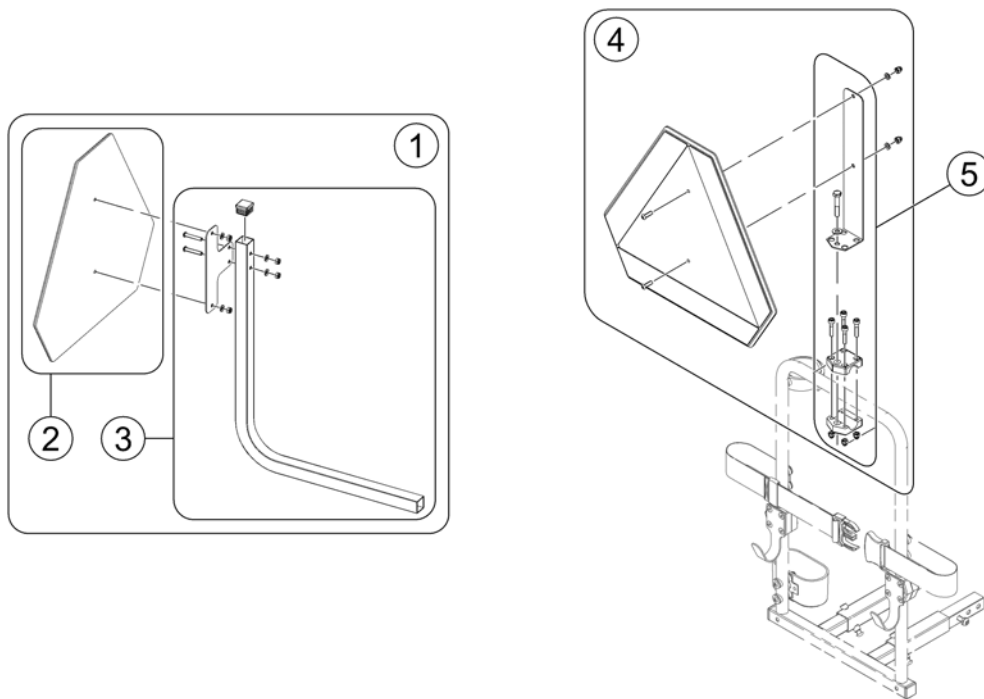

1	SP1523648	(Warning Triangle Scooter)	1
2	1430834	(Rear Marking Sign ECE-R 69)	1
3	SP1526913	(Holder Warning Triangel)	1
4	SP1430410	(Reflective Sign ECE-R 69, complete)	1
5	SP1580671	(Bracket, Reflective Sign, cpl.)	1

(Seating)


164017/00

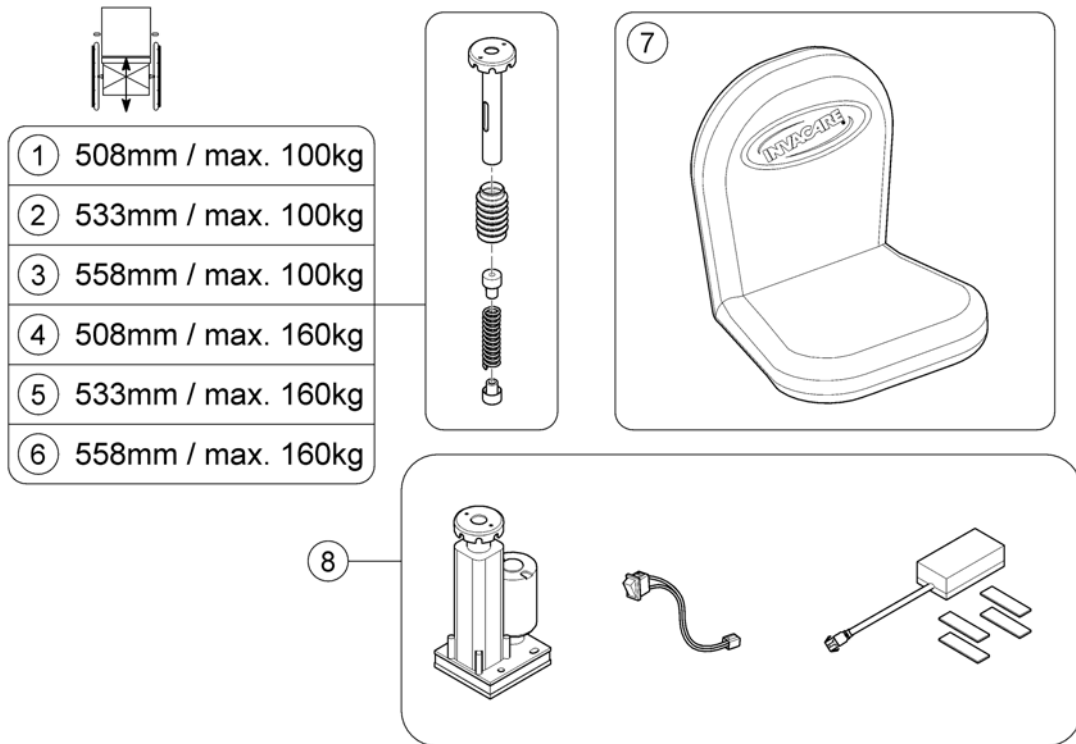

1	SP1640428	FJÆRING SETE SH50 COMET ALPINE+	1
2	SP1640429	(Seat Suspension 226mm 285lbs)	1
3	SP1640430	(Seat Suspension 251mm 201lbs)	1
4	SP1640447	(Seat Suspension 251mm 285lbs)	1
5	SP1640448	(Seat Suspension 276mm 201lbs)	1
6	SP1640449	(Seat Suspension 276mm 285lbs)	1
7	SP1521684	SETETREKK STD COMET <i>Not HD Seat!</i>	1
8	SP1641662	(Powered Seat Lifter, cpl.) <i>NOT Comet Ultra!</i>	1


(Walking support)

154030 no

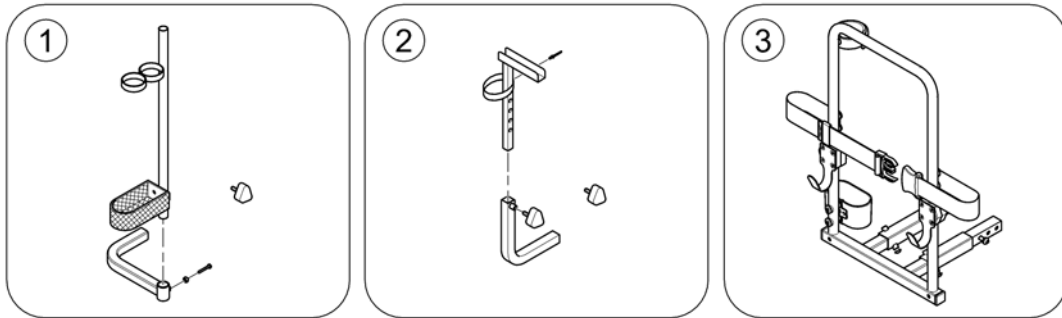


1	SP1528274	KRYKKEHOLDER COMET COMET	1
2	SP1505728	HOLDER FOR GÅRAMME COMET	1
3	-	(Rollator bracket)	0

(Rollator bracket)

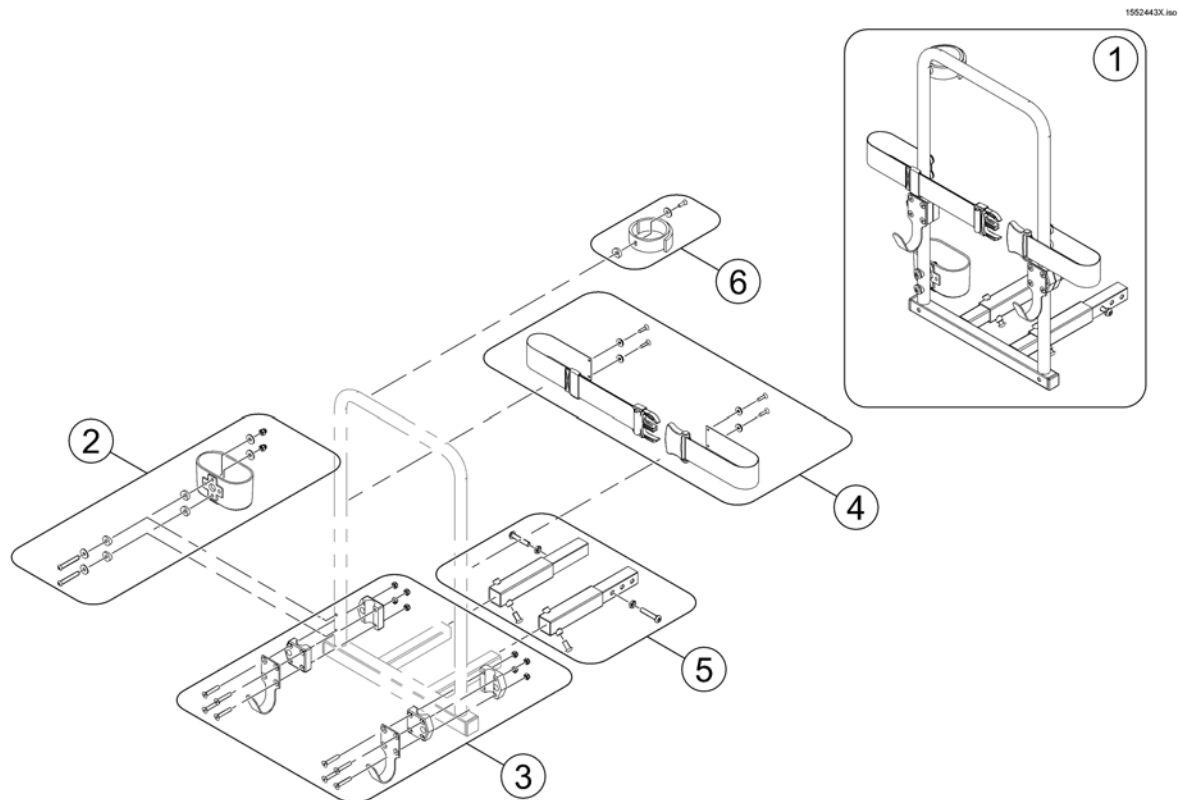


1	SP1553851	RULLATORHOLDER COMET UNI HVIS HD HUSK SP1552594	1
2	SP1560465	KRYKKEHOLDER KUN SKÅL COMET	1
3	SP1553890	(Rollator hooks)	1
4	SP1553891	(Belt, cpl.)	1
5	SP1552594	FORLENGER FESTE KRYKKEH COMET <i>only needed for Orion & Comet</i>	1
6	SP1640107	(Strap, crutch holder, cpl.)	1

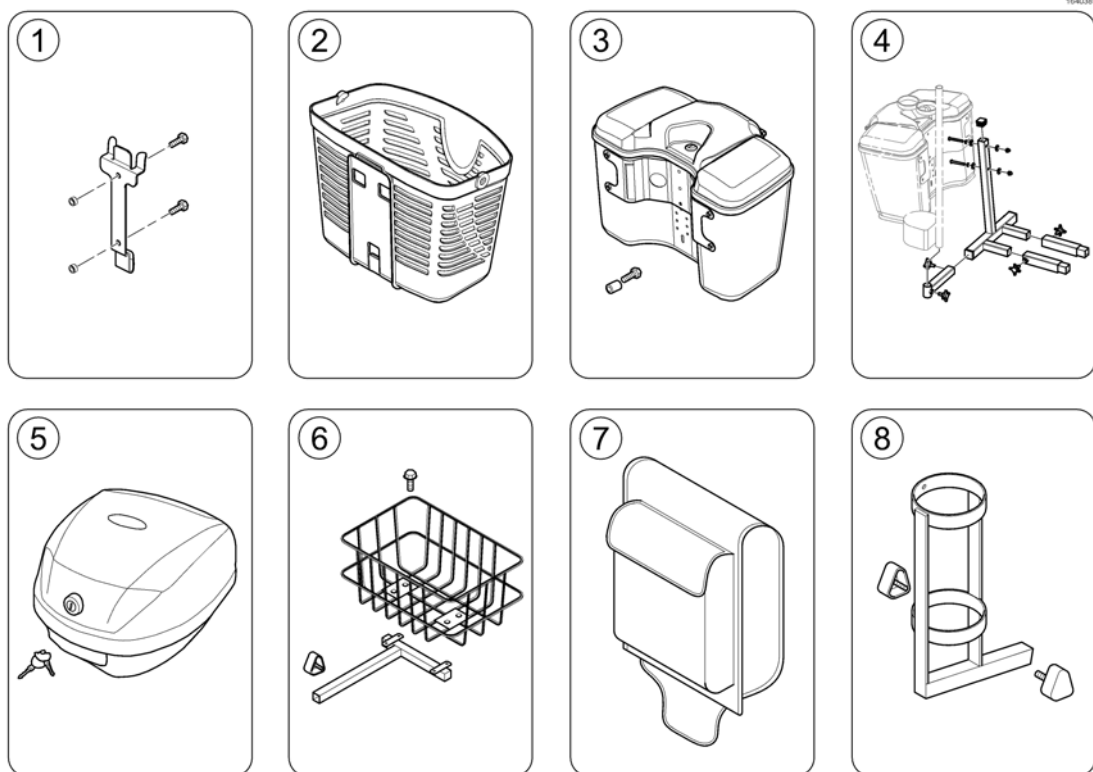(Storage)

1	SP1505924	(Bracket, Front Basket, cpl.)	1
2	SP1577430	KURV FORAN COMET+ ULTRA COMETA ALPINE+	1
2	SP1637915	KURV FORAN STOR COMET ALPINE+ <i>Not shown / Non illustré / Nicht abgebildet</i>	1
3	SP1637910	BAGASJEBOKS FORAN COMET ALPINE+	1
4	-	(Storage box rear bracket)	1
5	SP1637913	BAGASJEBOKS LÅSBAR COMET ALPINE+	1
6	SP1505727	KURV BAK COMET COMET	1
7	SP1505731	(Bag, Seat Rear)	1
8	SP1572971	HOLDER OKSYGENFLASKE COMET ALPINE+	1

(Storage box rear bracket)



1552537X.NO

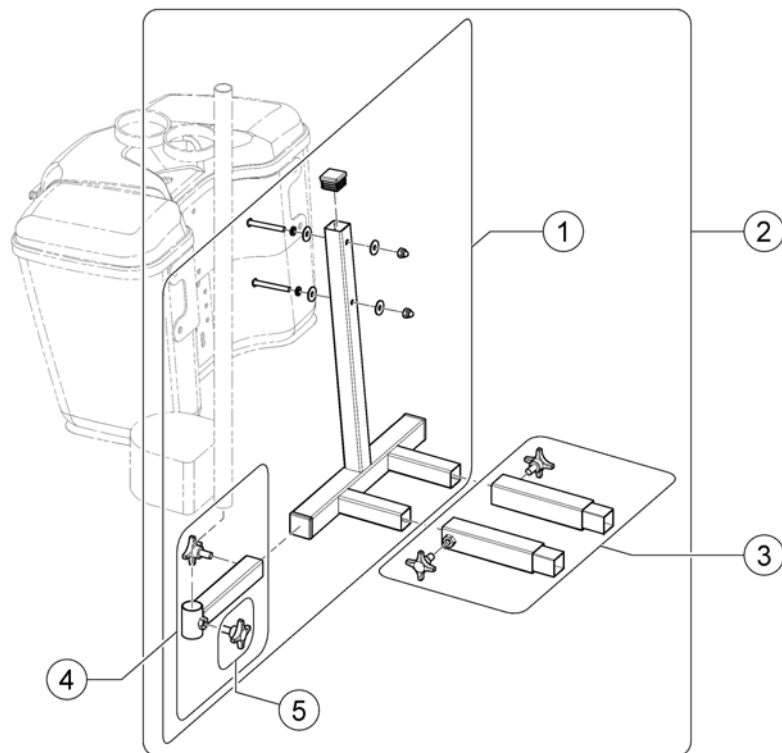

1	SP1552536	(Rear bracket for Storage box (Leo))	1
2	SP1552537	BRAKETT BAGASJEBOKS BAK COMET	1
3	SP1552594	FORLENGER FESTE KRYKKEH COMET	1
4	SP1553854	FESTE KRYKKEHOLDER COMET	1
5	5824117	STJERNEGREP FESTE NY MANØVERBOKS MOOVER	1


(Bumpers)

1640060 00

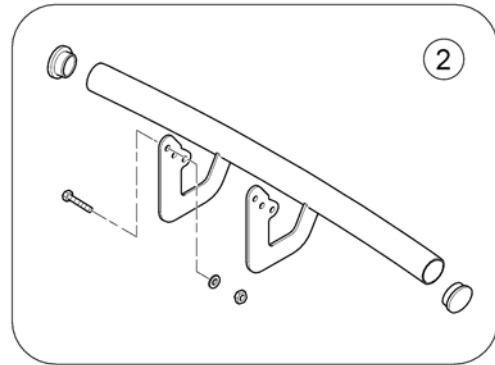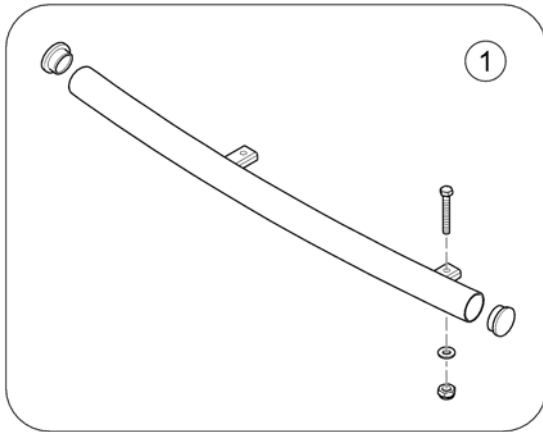

1	SP1638967	STØTFANGER BAK COMET ALPINE+	1
2	SP1528298	STØTFANGER FORAN COMET 4W	1

(Other Accessories)

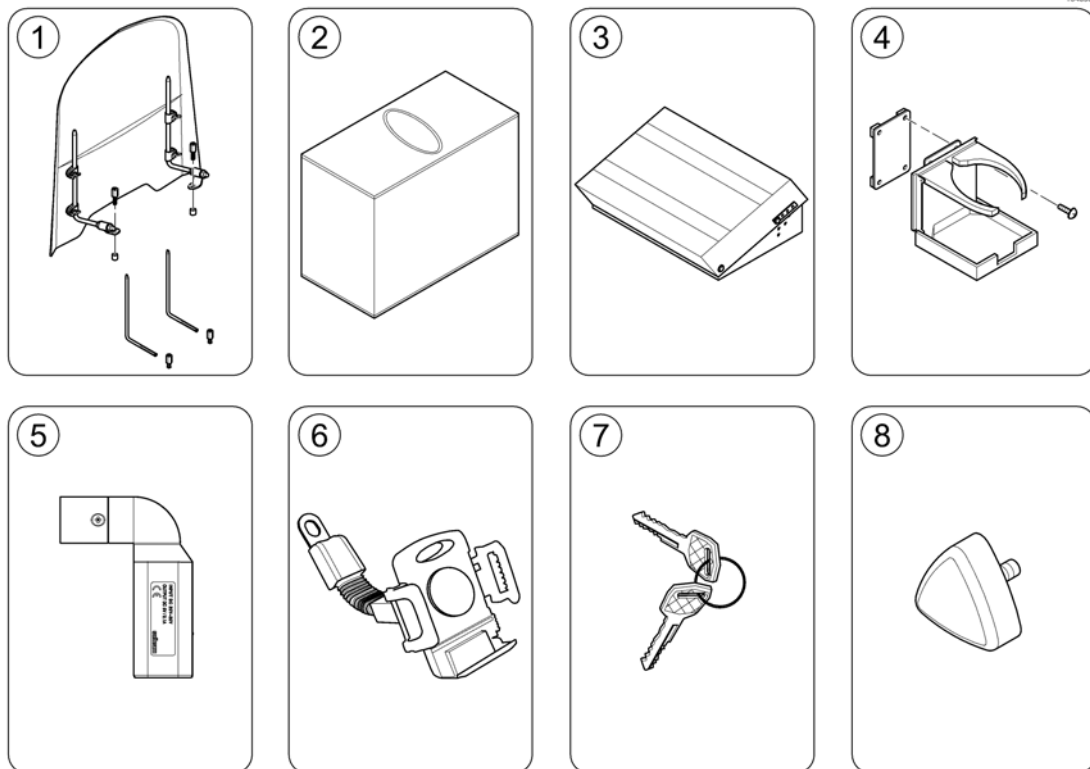

1	SP1543160	VINDSKJERM COMET	1
2	SP1518240	PRESENING COMET COMET	1
3	SP1637914	FOTSKAMMEL JUSTERBAR COMET ALPINE +	1
4	SP1521594	HOLDER KOPP COMET	1
5	SP1637917	USB LADEUTTAK FOR COMET COMET ALPINE+	1
6	SP1637912	HOLDER TELEFON COMET ALPINE+	1
7	SP1530211	NØKKEL 2 STK STANDARD COMET	1
8	SP1507005	(Star Handle)	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk