**NOTE:** Please read the following instructions carefully prior to assembly.

**This item should be fitted by an experienced clinician or technician. This document contains important information that must be passed on to the user at delivery.**

Before assessing for the positioning of the Anti-Thrust Harness, ensure that the user is sitting in an optimum position with their bottom as far back in the seat as is possible to assess for correct positioning.

### TRANSPORTATION

Invacare do not recommend the use of the Anti-Thrust Harness for use in transportation.

### MAINTENANCE

Before each use:

Undertake visual check for torn fabric and loose or worn screws.

Bi-Annually:

Ensure the Anti-Thrust Harness is inspected for torn fabric, loose or worn fastenings and that the torque settings are checked by a suitably qualified and competent person. For heavier and high impact users, the period between inspections should be reduced accordingly.

The Expected Product Life-Cycle for this product is 30 months.

### LIMITATIONS OF USE

Invacare do not recommend the use of the Anti-Thrust Harness for use in transportation.

The Anti-Thrust pelvic harness is suitable for use on most wheelchairs.

### TRAINING

If for any reason you should require additional training regarding the assembly of this hardware, please contact us at the number listed below, for free advice and support.

### COMPATIBILITY

The Anti-thrust Harness should only be used in conjunction with the Flo-tech PT Back Support system and a suitable Pelvic Belt.

### QUALITY

In keeping with our policy of continuous improvement, we reserve the right to modify designs without prior notice. All rights reserved.

### CLEANING AND CARE

The Anti-Thrust Harness may be cleaned with a neutral detergent (e.g. soapy hot water) and / or a single use wipe. The Nylon Covered Neoprene pads can be easily removed for machine washing at 60<sup>o</sup>c and should be air dried. Not suitable for tumble drying. Refer to washing label.

The mounting hardware may be cleaned with common household cleaners and a soft cloth. Should the product become contaminated, please ask your infection control officer for advice. Store in a dry environment.

### CAUTION

Proper installation of the Anti-Thrust Harness will prevent injury. A harness that is too loose will be ineffective and may allow excess movement and contact with surfaces that may cause harm. A harness that is too tight may result in restricted blood flow and cause discomfort. Not to be used as a restraint.

### Contraindications

- Hip dislocations, broken skin or irritation in the groin area.
- Please note this product may influence the effectiveness of a pressure redistribution cushion

**Anti-Thrust Harness**  
Qty: 1

**Universal Mounting Clamps**  
Qty: 2

### INSTRUCTIONS FOR ASSEMBLY

STEP 1—Remove Anti-Thrust Harness and components from the packaging.

STEP 2— Should this product reach you in an unsatisfactory condition, please contact the supplier.

STEP 3—When properly installed, the user will be sitting on the wide portion of the harness (see fig. 1). To install, lay the Anti-Thrust Harness on the wheelchair seat. The back of the harness should be at the back of the seat (see fig. 1). Attach the rear tri slot mounting plates (see fig. 2) into the desired position on the wheelchair frame using the Universal Mounting Clamps, which are suitable for round tube frames of 19mm, 22mm, and 25mm (see fig. 3).

STEP 4 – Position the user on the harness with the front padded straps wrapping over the legs (see fig. 4). Attach the front tri slot mounting plates on the front straps of the harness into the desired position on the wheelchair frame using the Universal Mounting Clamps, which are suitable for round tube frames of 19mm, 22mm, and 25mm (see fig. 3).

**It is recommended that all screws be tightened to 8.5Nm.**

**Figure 1**

**Figure 2**

**Figure 3**

## INSTRUCTIONS FOR USE

The Anti-Thrust Harness may be easily removed and re-installed. Each strap has a quick release buckle to detach the harness from the chair (see fig. 6). To remove the harness squeeze the buckle with thumb and fore finger and pull the strap out. Repeat with second strap on opposite side to transfer the user or all four straps to remove the harness completely.

### Adjusting the straps

Each strap is adjustable (see figure 6). When the strap is pulled, it tightens the harness around the user. Repeat at each strap until harness is snug.

Figure 4

Figure 5

Figure 6

Figure 5

## WARRANTY

The Flo-tech PT Anti-Thrust Harness is covered by a warranty period of 12 months.

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