Invacare® H450LA Omega

| | | |
|----|--|----|
| en | Commode chair User Manual | 3 |
| de | Toilettenstuhl Gebrauchsanweisung | 5 |
| fr | Chaise toilette Manuel d'utilisation | 7 |
| it | Sedia comoda Manuale d'uso | 10 |
| es | Silla de inodoro Manual del usuario | 12 |
| pt | Cadeira sanitária Manual de utilização | 14 |
| nl | Toiletstoel Gebruiksaanwijzing | 16 |
| da | Toiletstol Brugsanvisning | 18 |
| no | Toalettstol Bruksanvisning | 20 |
| sv | Hygienstol Bruksanvisning | 22 |
| fi | Kiinteä suihkutuoli Käyttöohje | 24 |
| cs | Židle na toaletu Návod k obsluze | 26 |
| hr | Toaletna stolica Upute za uporabu | 28 |
| et | Tualett-tool Kasutusjuhend | 30 |
| lt | Naktipuodžio kėdė Naudojimosi instrukcija | 32 |

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

Images

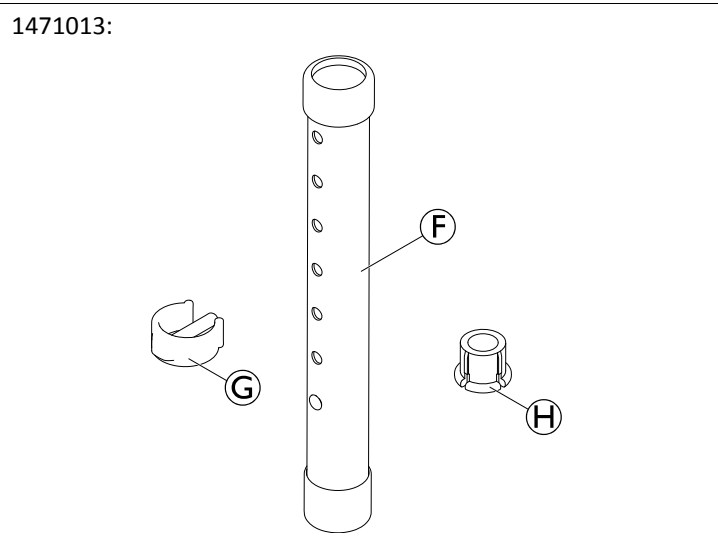
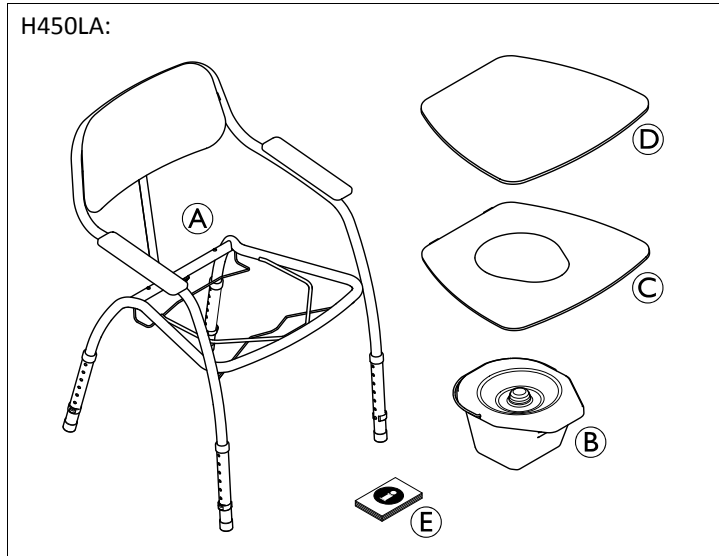
Fig. 1

Fig. 2

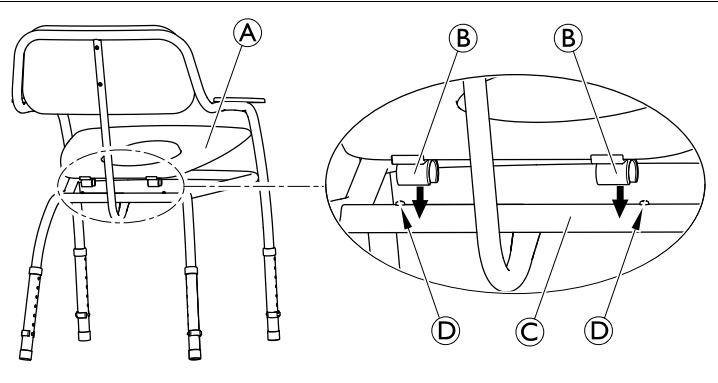
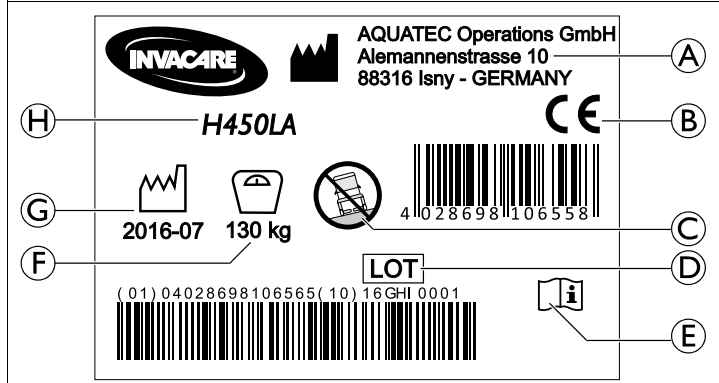
Fig. 3

Fig. 4

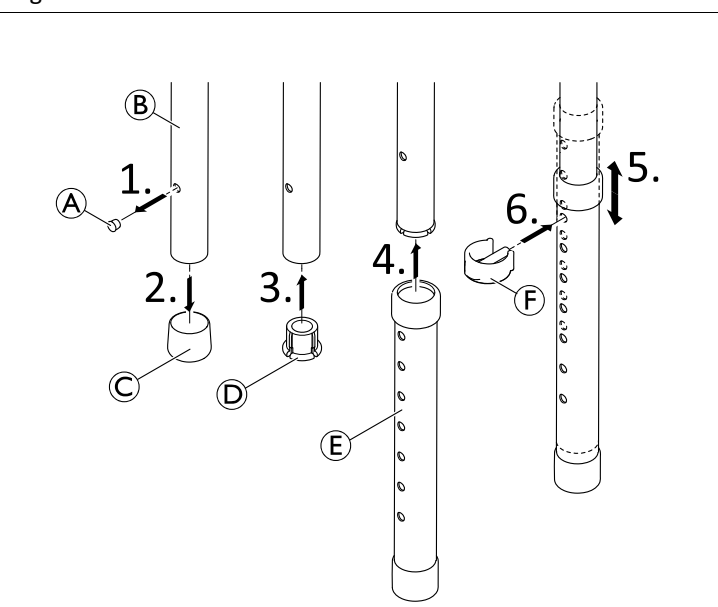
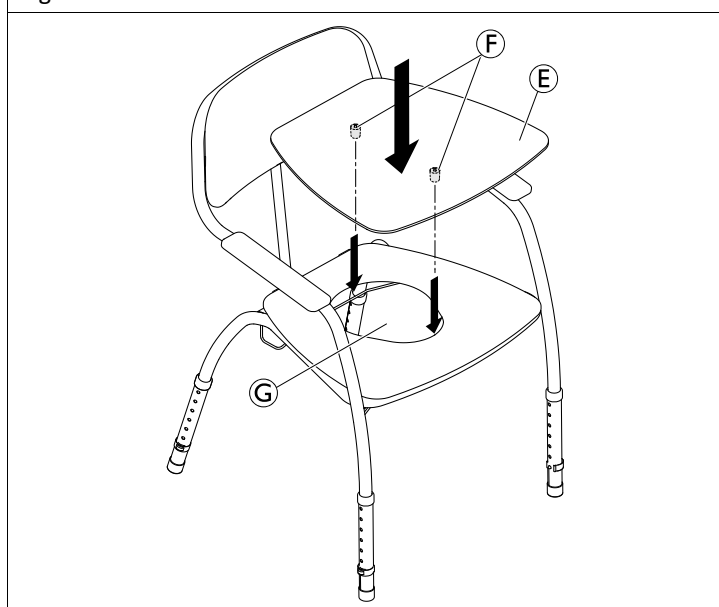
Fig. 5

Fig. 6

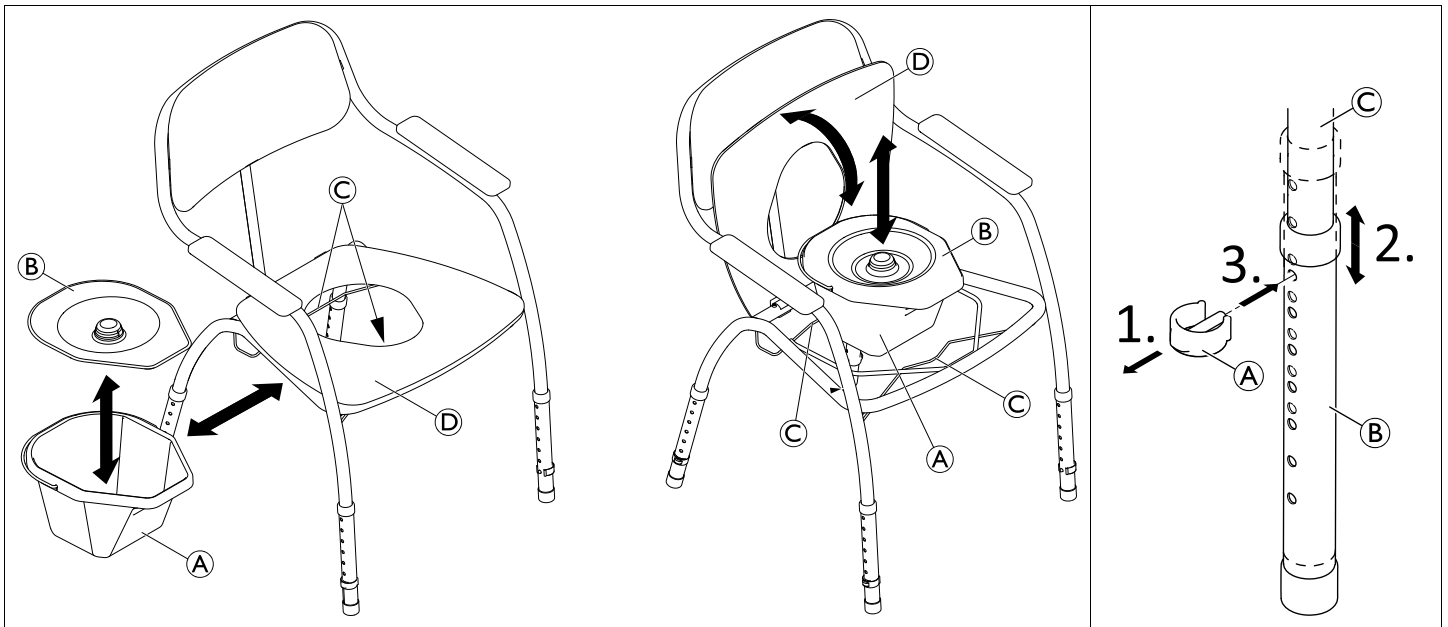
Fig. 7

Fig. 8

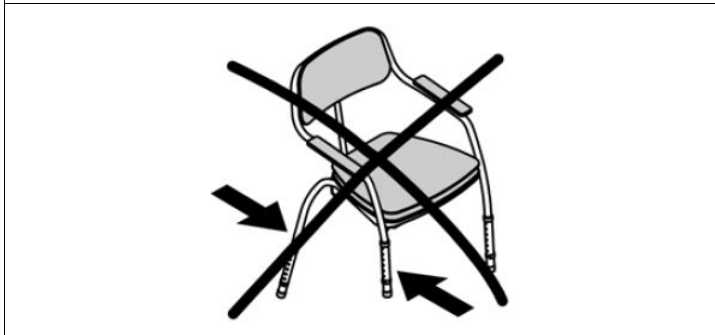
Fig. 9

Fig. 10

General

en

Important note:

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required. If you cannot enlarge the text and graphics sufficiently, please contact the Invacare distributor for your country. For addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

Intended use

The product is intended to be used solely as a commode in a indoor environment by adult people who have:

- Difficulty standing or keeping their balance
- Weak legs or lower limb disability
- Medical conditions that impair their balance

Any other use is prohibited.

Operating temperature: 10 - 40 °C

Max. user weight: 130 kg

Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see back page of this manual).

Warranty


We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

Scope of delivery

H450LA (Fig. 1)

| | |
|-----|-----------------------------------|
| (A) | Fixed commode chair with armrests |
| (B) | Toilet pan with handle and cover |
| (C) | Toilet seat |
| (D) | Removable seat cover plate |
| (E) | User manual |

1471013 (Fig. 2)

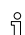 The retrofit kit 1471013 is already assembled in the scope of delivery of H450LA.

| | |
|---|-----------------------------|
| Ⓕ | Adjustable tube (4x) |
| Ⓖ | Height adjustment clip (4x) |
| Ⓗ | Guide bushing (4x) |

Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety**Safety information****WARNING!****Risk of injury!**

- Do not use if defective.
- Do not use in the shower (Fig. 10).
- Keep product away from open flames.
- Do not place smoldering cigarettes on the product.
- Only use spare parts and accessories intended for this product.
- Do not make any unauthorized alterations or modifications to the product.

**WARNING!****Risk of falling!**

- Do not stand on this product.
- Do not use as a climbing aid.
- Do not lift the product at the armrest for re-positioning or transfer while seated.
- Do not pull/push the chair to displace it under load.

**WARNING!****Risk of burning your skin!**

- The product components can heat up when exposed to external sources of heat.
- Do not expose the product to strong sunlight before use.

Symbols on the product (Fig. 3)

The identification label is located on the bottom of the seat.

| | |
|---|---|
| Ⓐ | Manufacturer address |
| Ⓑ | This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity. |
| Ⓒ | Only use on flat surfaces |
| Ⓓ | Lot number |
| Ⓔ | Read user manual |
| Ⓕ | Maximum user weight |
| Ⓖ | Date of manufacture |
| Ⓗ | Product designation |

Assembly**Toilet seat (Fig. 4)**

1. Attach the toilet seat Ⓐ with the clips Ⓑ to the rear frame tube Ⓒ.

**CAUTION!**

- Make sure that both clips engage with an audible click.
- Make sure that both clips are placed between the stop screws Ⓓ and not on top of the stop screws.

Seat cover plate (Fig. 5)

1. If desired, insert the seat cover plate Ⓔ from above into the toilet seat opening.

**IMPORTANT!**

- Make sure that both stoppers Ⓕ on the underside of the seat plate engage into the opening Ⓖ of the toilet seat.

Adjustable leg tubes (Fig. 6) (Retrofit kit 1471013)

1. Remove cap Ⓐ from the frame tube Ⓑ.
2. Remove rubber tip Ⓒ from the end of the frame tube.
3. Insert guide bushing Ⓓ into the frame tube.
4. Push the adjustable tube Ⓔ over the frame tube.
5. Position the adjustable tube to the desired height.
6. Insert the height adjustment clip Ⓕ into the holes of the adjustment tube and frame tube.

**WARNING!****Risk of injury**

- Make sure that the clip fully engages through both holes with an audible click.

7. Repeat the same steps for the remaining three legs.

**WARNING!****Risk of injury**

- Make sure that all legs are adjusted to the same height (Fig. 9).

Usage**Adjusting the height of the legs (Fig. 8)**

1. Remove the height adjustment clip Ⓐ from the holes of the adjustment tube Ⓑ and frame tube Ⓒ.
2. Position the adjustable tube to the desired height.
3. Re-insert the height adjustment clip into the holes of the adjustment tube and frame tube.

**WARNING!****Risk of injury**

- Make sure that the clip fully engages through both holes with an audible click.

4. Repeat the same steps for the remaining three legs.

**WARNING!****Risk of injury**

- Make sure that all legs are adjusted to the same height (Fig. 9).

Inserting/Removing the toilet pan (Fig. 7)

The toilet pan Ⓐ can be inserted/removed with or without cover Ⓑ.

1. Insert the upper rim of the toilet pan Ⓐ from the side between the toilet seat Ⓓ and the brackets Ⓒ under the seat.

or (in case the user is not sitting on the commode)

1. Fold up the toilet seat ①.
2. Place the toilet pan ② from above between the brackets ③.

 To remove the toilet pan, repeat steps 1 and 2 in reverse order.

 The toilet pan can be lifted and carried by its handle.

Maintenance and periodic inspection

The product is maintenance-free, provided that the cleaning and safety instructions outlined in this user manual are adhered to.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled. Make sure that all screws are securely tightened.
- Regularly clean and disinfect the product.

Cleaning and disinfection

 The product can be cleaned and disinfected using commercially available agents.

1. Clean the product regularly.
2. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

IMPORTANT!

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.
 - Clean by hand.
 - Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
 - Max. cleaning temperature 60 °C.

Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly, check the product for damages and ensure a proper mounting.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

Materials

| | |
|---|---------------------|
| Frame | ST37, powder coated |
| Upholstery (Seat / backrest / armrest pads) | PVC |
| Toilet pan | PP |

All product components are corrosion resistant and do not contain natural rubber latex.

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Invacare Poirier SAS
Invacare Austria GmbH
Invacare AB
Aquatec Operations GmbH