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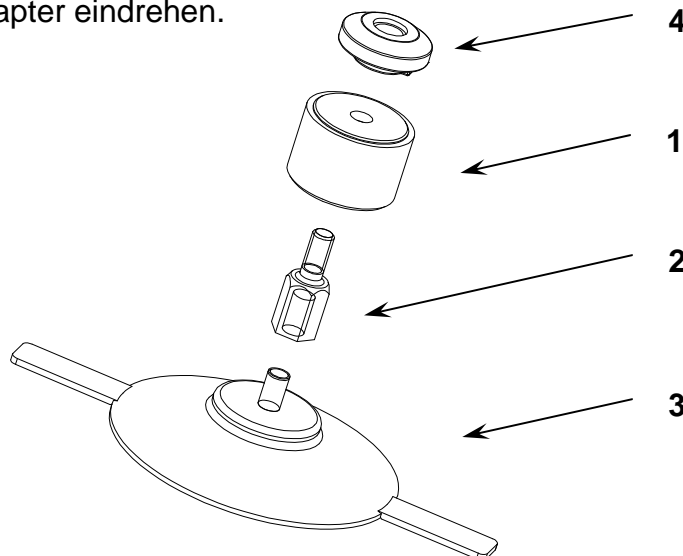
Please read carefully the complete assembly instructions before you start the assembly!

## **A. ACCESSORIES**

1. Height adapter 2 cm (4 pieces)
  - incl. conversion kit order no. 15170
  - extension kit without conversion kit order no. 14694
2. Assembly instructions (1 page)

## **B. ASSEMBLY OF THE FRONT HEIGHT ADAPTERS**

1. Take out the suction cup lid from the base plate and remove the suction cup with toggle by pulling it backwards (cannot be used anymore).
2. Replace the suction cup lid in the keyhole bore of the base plate.
3. Place the adapter insert (4) in the keyhole bore so that the nose on the bottom side matches with the groove at the base plate.
4. Screw the height adapter (1) with the threaded bolt (2) from below into the thread on the adapter insert (4).
5. Screw the new suction cups (3) into the height adapter.

## **C. ASSEMBLY OF THE REAR HEIGHT ADAPTERS**

1. Unscrew the suction cups from the base plate of the bathlift.
2. Screw in the height adapters (1) with corresponding threads (2) from below into the thread of the base plate.
3. Screw the suction cup (the one you unscrewed) into the height adapter.