# Invacare® Matrix® PS

en	<b>Cushion</b>	User Manual . . . . .	1
de	<b>Kissen</b>	Gebrauchsanweisung . . . . .	4
fr	<b>Coussin</b>	Manuel d'utilisation . . . . .	7
it	<b>Cuscino</b>	Manuale d'uso . . . . .	10
es	<b>Cojín</b>	Manual del usuario . . . . .	13
pt	<b>Almofada</b>	Manual de utilização . . . . .	16
nl	<b>Kussen</b>	Gebruiksaanwijzing . . . . .	19
da	<b>Pude</b>	Brugsanvisning . . . . .	22
no	<b>Pute</b>	Bruksanvisning . . . . .	25
sv	<b>Dyna</b>	Bruksanvisning . . . . .	28
fi	<b>Pehmuste</b>	Käyttöohje . . . . .	30

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

## 1.1 General information

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions. For further information contact Invacare® in your country (addresses see back page of this manual).

## 1.2 Symbols in this user manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

### WARNING

Indicates a potentially hazardous situation which if not avoided could result in death or serious injury.

### CAUTION

Indicates a potentially hazardous situation which if not avoided could result in product damage, minor injury or both.

### IMPORTANT

Indicates a hazardous situation which if not avoided could result in damage to the product.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

## 1.3 Warranty

We provide a manufacturer's warranty in accordance with our General Terms and Conditions of Business in the respective countries. Guarantee claims can only be made through the provider from whom the appliance was obtained.

## 1.4 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485 and ISO 9001.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We use only REACH compliant materials and components.

The foam and cover used to manufacture the product are fire safety tested and certified in accordance with EN 1021-1 & -2.

For further information please contact Invacare® in your country (addresses see back page of this manual).

## 1.5 Intended use

The Invacare Matrx PS is designed to provide superior positioning, stability and comfort. Normal use is defined by Invacare as when the product is placed without additional covers or padding between the user and the support surface.

## 1.6 Product description

The Invacare® Matrx® PS cushion is designed to provide skin protection and positioning for individuals at moderate to high risk of skin breakdown. The Invacare® Matrx® PS cushion features an anatomically contoured shape with a unique waffled ischial relief recess and an innovative reversible outer cover. Our standard outer cover utilizes a unique reversible Startex™ Fabric, which provides two unique cover options to satisfy your needs in terms of comfort and aesthetics. This specially designed, reversible stretch fabric outer cover may be installed with either the knitted surface exposed or with the smooth surface exposed. An non-reversible Kidabra fabric cover is also available for paediatric users. This optional Kidabra cover is available on all our paediatric cushion sizes. All outer cover options

are moisture resistant and breathable. The inner cover is moisture resistant to protect the foam.

## 2 Safety

### 2.1 Safety information

#### WARNING!

– Do not use this product or any available optional equipment without first completely reading and understanding the user manual supplied. Invacare® product manuals are available on your local Invacare website or at your local dealer. If you are unable to understand the warnings, cautions or instructions please contact a health care professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

#### WARNING!

Invacare® products are specifically designed and manufactured for use in conjunction with Invacare® accessories. Accessories designed by other manufacturers have not been tested by Invacare® and are not recommended for use with Invacare® products.

#### IMPORTANT!

The information contained in this document is subject to change without notice.

- Check all parts for shipping damage and test before using.
- In case of damage, do not use.
- Contact Invacare® / Carrier for further instructions.

### 2.2 Symbols on the product

	Refer to user manual		Do not bleach
	CE conform		Do not iron
	Tumble dry low heat		Machine wash temperature max. 60 °C
	Do not dry clean		

## 3 Setup

### 3.1 Installing the cushion

1. Remove all packaging before installation.
2. Ensure the desired side of the outer cover is exposed. Refer to section "Reversing the outer cover" below.
3. Position the cushion so the material identification tags (not shown) face the rear.
4. Align the hook and loop strips (not shown) on the bottom of the cushion with those on the wheelchair seat.
5. In the event that the wheelchair seat and PS Cushion have matching loop strips, install the four double hook strips (provided separately) at each of the intersection points on the wheelchair seat to secure the cushion in place.
6. Position the cushion onto the wheelchair as far back as possible.
7. Press down to ensure the hook and loop strips are firmly engaged.

## Reversing the outer cover

- !** **IMPORTANT!**  
Unlike the standard Startex™ fabric, the Kidabra Cover is non-reversible  
– Do not use the Kidabra Cover turned inside out.

1. Unzip the outer cushion cover.
2. Remove the outer cushion cover.
3. Turn the outer cushion cover inside out to the desired side.
4. Install the cushion with inner cover into the outer cover with the waffled recess facing down.

## 4 Usage

### 4.1 Safety information

- !** **WARNING!**  
**Risk of injury**  
– Installing a cushion on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to become unstable, potentially resulting in injury.  
– After any adjustments, repair or service and before use, make sure that all attaching component parts are secure.  
– Ensure the seat cushion is properly secured to the wheelchair before using. Otherwise injury or damage may occur.  
– Make sure that the locking clips are installed to ensure the safety of the user and to prevent the cushion from lifting.

- !** **WARNING!**  
**Risk of injury**  
– Skin condition should be checked very frequently after the installation of any new seating system.  
– Your therapist and physician should be consulted if you have any questions regarding individual limitations and needs.  
– Working with your therapist, physician, and equipment supplier is the best way to ensure that a seating choice matches your individual needs.  
– As the needs of the individual become more complex, the seating evaluation becomes more important.  
– The cushion is designed to be used only at room temperature. If it is left at lower temperatures, always allow it to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.  
– Avoid sharp objects or exposure to excessive heat or open flame.

## 5 Maintenance

### 5.1 Inspection

- !** **WARNING!**  
– Do not continue to use this product if problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

1. Visually inspect parts including drop hooks, hardware, upholstery materials, foams (if accessible), and plastics, for deformation, corrosion, breakage, wear or compression and replace if necessary.

## 5.2 Cleaning and care

- !** **IMPORTANT!**  
All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.  
– For further information on decontamination in health care environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CGI39](http://www.nice.org.uk/CGI39) and your local infection control policy.

### Cleaning the outer Startex™ cover

1. Wipe clean the outer cover with a damp cloth.  
or
2. Carefully remove the cover by unzipping the zipper found on the rear of the cushion.
3. Close the zipper prior to laundering.
4. Machine wash at 60° C using the delicate cycle (Instructions on label).

- !** **IMPORTANT!**  
Washing at higher temperatures will cause shrinkage.

### Drying the outer Startex™ cover

1. Hang cover from a line or bar and drip dry in a clean indoor environment.  
or  
Tumble dry on a low heat setting.

- !** **IMPORTANT!**  
– Tumble dry setting must not exceed 40 °C.  
– Do not tumble dry for longer than 10 minutes.  
– Dry thoroughly before re-fitting to the foam.

### Cleaning the inner cover

- !** **IMPORTANT!**  
**Risk of damage to the product**  
– Do not remove the inner cover from the foam core.

1. Wipe clean the inner cover with a damp cloth.

### Replacing covers

1. Carefully remove the cover by unzipping the zipper found on the rear of the cushion.
2. Place new cover onto the foam core.

- !** **IMPORTANT!**  
– Ensure that the corners of the foam core are positioned correctly into the corners of the cover.

## 6 After Use

### 6.1 Storage

- !** **IMPORTANT!**  
– Store cushions in a dry environment.  
– Store cushions within a protective cover.  
– Store items on clean, dry, off-flooring free from sharp edges to avoid any possible damage.  
– Never store other items on top of a cushion.  
– Do not store cushions next to radiators or other heating devices.  
– Protect cushions from direct sunlight.

1. Refer to the storage and shipping conditions in section 7.3 Environmental Parameters, page 4.

### 6.2 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used.

**WARNING!**  
**Risk of injury**

– Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, do not use the product.

- I. Before reuse, clean the product thoroughly, see section 5.2 Cleaning and care, page 3.


**6.3 Disposal**

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

- I. Ensure that the cushion is cleaned prior to disposal to avoid any risk of contamination.

**7 Technical data**

**7.1 General data**

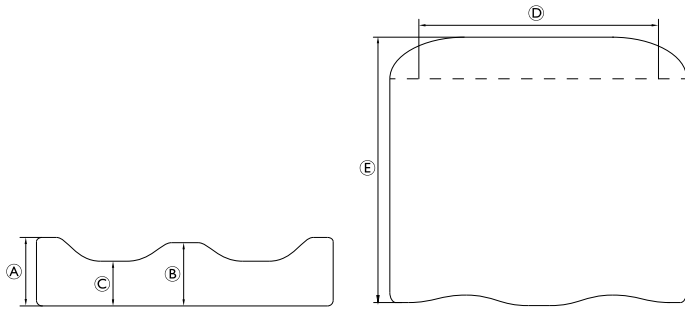Matrx® PS	Paediatric	Standard	Heavy duty
Adductor height (A) [mm]	95	102	102
Abductor height (B) [mm]	89	89	89
Leg trough height (C) [mm]	64	64	64
Seat width (D) [mm]	254 / 279 / 305 / 330 / 356	406 / 457 / 508	533 up to 762
Seat depth (E) [mm]	254 / 305 / 356 / 406	406 / 457 / 508	457 up to 762
Maximum user weight [kg]	68	136	272
Product weight [kg] <sup>1)</sup>	0.5	0.9	3.2

<sup>1)</sup> Weights can vary depending on size of cushion ordered.

Size charts and local requirements / certification available upon request from Invacare.

**7.2 Materials**

Foam	Moulded High Resilience (HR) Polyurethane Foam
Inner cover	Reversed Startex™ - 100% Interlock Polyester (black) with Polyurethane Coating (coated side up)
Reversible outer cover	Startex™ - 100% Interlock Polyester with Polyurethane Coating
(Optional) Paediatric Outer Cover (Non-Reversible)	Kidabra fabric - 100% Polyester Mesh with Clear Urethane laminate backing
All cushion components are not made with natural rubber latex.	

**7.3 Environmental Parameters**

Operating conditions	
Ambient temperature	10 - 35 °C
Relative humidity	30% - 75% Non-Condensing
Atmospheric pressure	70 - 106 kPa
Storage and shipping conditions	
Ambient temperature	-40 - 70 °C
Relative humidity	10% - 100%
Atmospheric pressure	50 - 106 kPa

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Invacare AG
Advena Ltd.