
Invacare® Matrix® Libra®

en	Cushion accessory kit User Manual	1
de	Kissen-Zubehör-Kit Gebrauchsanweisung	4
fr	Kit d'accessoires du coussin Manuel d'utilisation	5
it	Kit di accessori del cuscino Manuale d'uso	6
es	Kit de accesorios de cojines Manual del usuario	7
pt	Kit de acessórios de almofada Manual de utilização	8
nl	Accessoireset voor kussen Gebruiksaanwijzing	9
sv	Tillbehörspaket för kudde Bruksanvisning	10
no	Tilbehørsett for pute Bruksanvisning	11
da	Tilbehørssæt til pude Brugsanvisning	12
fi	Pehmusteiden lisävarustesarja Käyttöohje	13




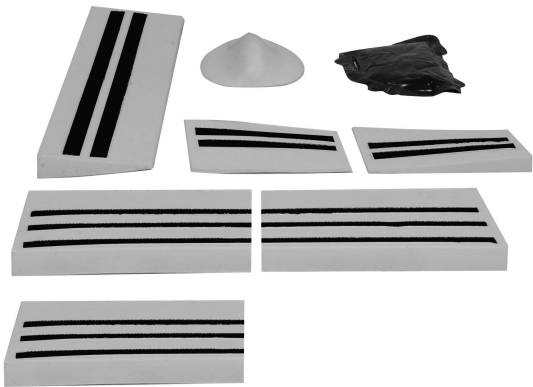
This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

I.1 Images

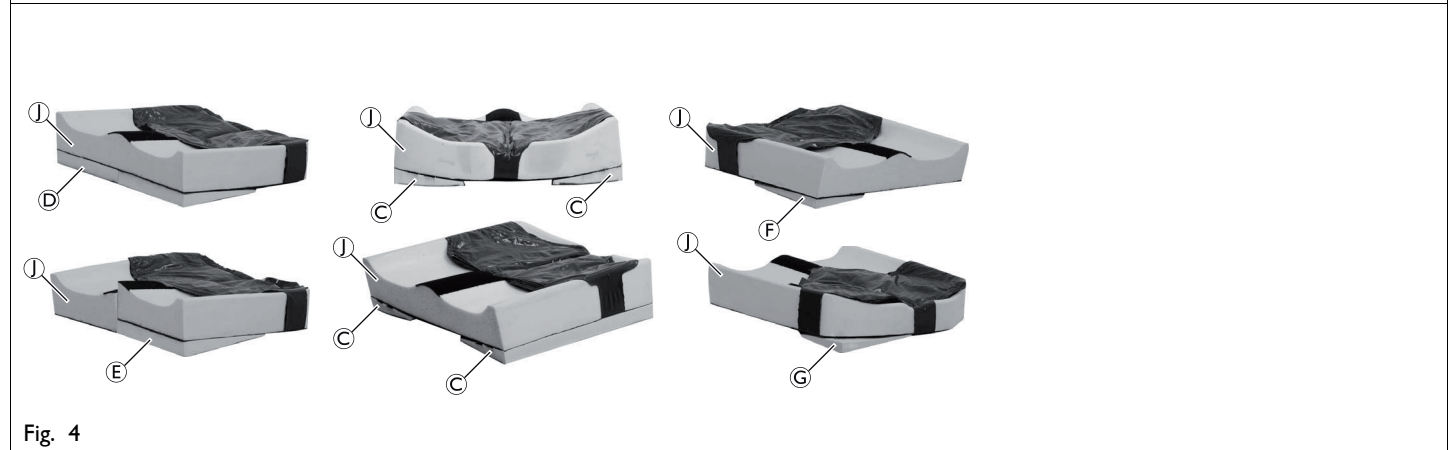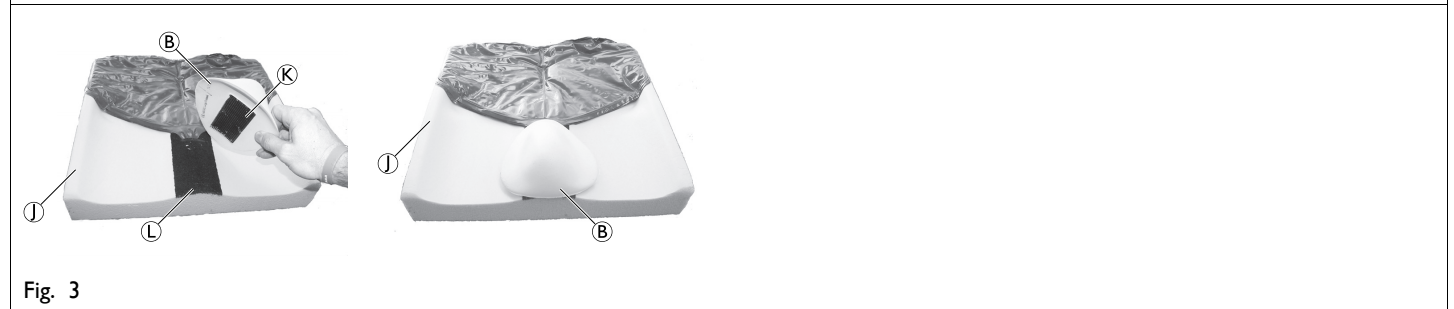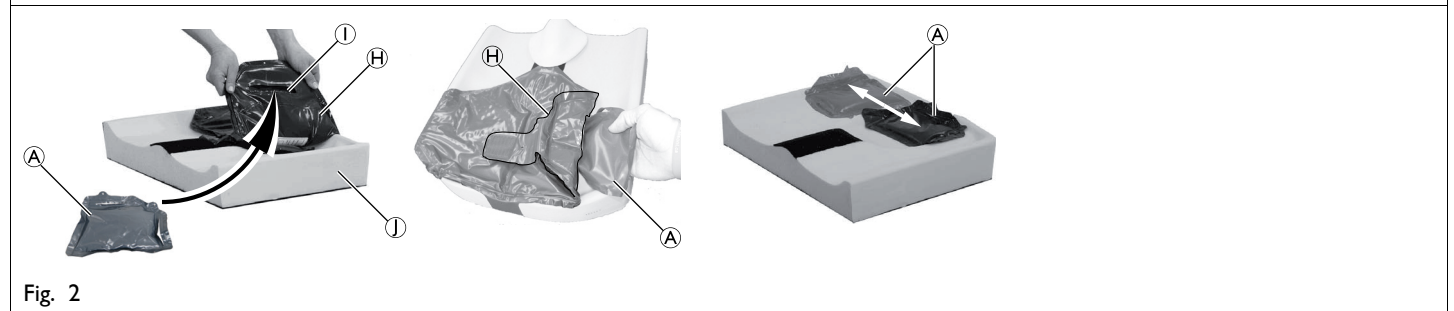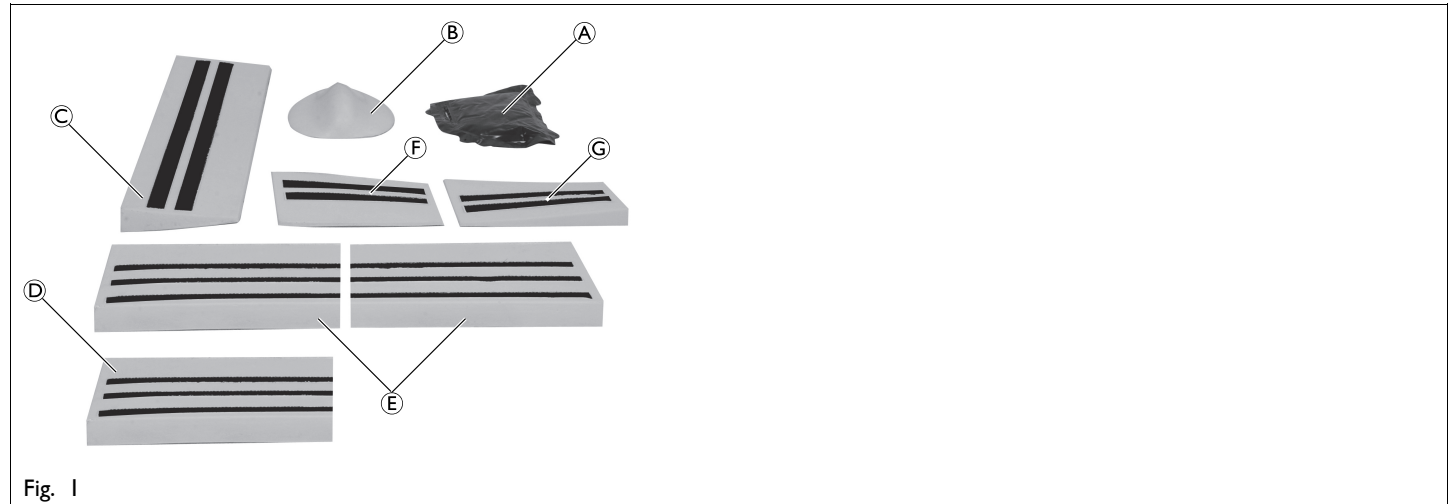1.2 Intended use

IMPORTANT!

- Read sections General, Safety, Maintenance and After use contained in the Invacare® Matrx® Libra® cushion User Manual in conjunction with this document.

Invacare® Matrx® Libra® accessories are intended for specific positioning needs of the end user. Any other use is prohibited.

1.3 Scope of delivery (Fig. 1)

The following components are included in the accessory kit:

Ⓐ	Fluid supplement sac
Ⓑ	Abductor
Ⓒ	Lateral side wedge
Ⓓ	Cut to size anterior (front) wedge
Ⓔ	Anterior (front) wedge
Ⓕ	RF/LR tapered wedge
Ⓖ	LF/RR tapered wedge

2 Setup

2.1 Installing the fluid supplement sac (Fig. 2)

The fluid supplement sac utilizes Flo-tech™ technology and non-temperature sensitive fluid. It can be easily placed in the pelvic well and/or on the trochanteric shelf for targeted skin protection around bony prominences.

1. Remove outer cover and inner liner.
2. Unfasten the hook and loop strips from the side and/or rear of the main fluid sac ④ where you wish to install the fluid supplement sac(s) ①.
3. Thread the desired strap from the main fluid sac through the slot on the fluid supplement sac, placing it between the main fluid sac and foam base ①.
4. Alternatively, locate the small pocket ① on the bottom of the main fluid sac and install the fluid supplemental sac inside the pocket. Adjust the fluid supplement sac to the desired position inside the main fluid sac.
5. Refasten the fluid sac hook and loop strips and re-install the inner liner and outer cover.

2.2 Installing the abductor (Fig. 3)

For comfortable mid-line positioning of thighs and hips.

1. Remove the outer cover and inner liner.
2. Align the hook strip ④ on the abductor support ③ with the loop strip ⑤ on the top of the foam base ① at the desired depth.
3. Press down to secure the abductor support to the foam base.
4. Re-install the inner liner and outer cover.

2.3 Installing the anterior, lateral and tapered wedges (Fig. 4)

The Anterior wedge increases slope, prevents sliding, and reduces shear. Can be trimmed to size as needed (see separate instructions). Lateral wedges reinforce lateral stability for enhanced midline positioning. Tapered wedges enhance adductor support when used in the front and can address obliquity and pelvic rotation issues when used in the rear.

1. Remove the outer cover and inner liner and position the wedge(s) in the desired position on the underside of the foam base ① as shown:
 - Lateral wedge ③
 - Anterior wedge ⑤ and cut to size anterior wedge (For example, hip flexion limitations and/or lower seat height needs for foot propulsion.)
 - RF/LR tapered wedge ⑥
 - LF/RR tapered wedge ⑦
2. Using the hook and loop strips, secure wedge(s) to the underside of the foam base.
3. Re-install the inner liner and outer cover.

2.4 Trimming accessories and/or foam base

Components can be trimmed to provide tailored, individual solutions for the end user. The anterior and lateral wedges require trimming to fit the foam base. Use a cutter or scissors to trim these accessories.

CAUTION!

Risk of cutting!

- Take care when using a cutter or scissors.

1. Trim the base of the cushion to accommodate leg length discrepancies, limited joint movement and/or contractures. Refer to Fig. 5 as an example.

3 Cleaning and care

The Invacare® Matrx® Libra® cushion and accessories should be stored in a clean, dry place. Accessories can be surface cleaned using mild detergent and water.

Also refer to the cleaning instructions in the Invacare® Matrx® Libra® cushion User Manual.

4 Materials

Foam accessories	High resilient Polyurethane foam
Fluid supplement sac	Polyether based thermoplastic polyurethane film
Backing loop	Polyester
Hook base and hook	Polyamide
All product components are Latex-free.	

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Invacare New Zealand Ltd
Invacare UK Operations Limited