

Spare Parts Catalogue

Version: 2015/08/13d EN





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.[®]

TABLE OF CONTENTS

Information & User Manuals

User Manuals Kite	9
User Manuals REM24SD	10
User Manuals REM A & REM B	11
User Manuals REM550	12
User Manuals G90	13
User Manuals G91	14
User Manuals REM34 & REM41 Tray/Centre Control	15
User Manuals IDC Dual Control	16
User Manuals Compact Dual Control	17
User Manuals Headcontrol	18
User Manual Proximity Head Remote DSL-105	19
User Manuals iPortal	20
User Manuals ACS Handprogrammer	21

Chassis

Battery box	22
DST-Bearing	23
Frame	24
Suspension	25
Covers	26
Anti-Tipper	27
Anti-Tipp-Wheels for Lifter	28

Seat

Seat	29
Standard Seat	30
Tilt 12°	31
Tilt 12° HD	32
Tilt 28°	33
Lifter	34
Seat Frame & Plate Standard	35
Seat Cushion	36
Flex2 Seat	37
Tilt 12°	31
Tilt 12° HD	32
Tilt 20°	38
Tilt 28°	33
Lifter	34
Seat Frame with Plate	39
Seat cushion	40
Max Seat	41
Tilt 12°	31
Tilt 12° HD	32
Tilt 20°	38
Tilt 28°	33
Lifter	34
Seat Frame with Plates	42
Seat Cushion Max	43
Modulite Seat	44



Yes, you can.®

TABLE OF CONTENTS

Tilt 12°	45
Tilt 20°	46
Tilt 30°	47
Lifter	48
One Piece Seat Plate	49
Telescopic Seat Frame and Plate	50
Telescopic Seat Frame for Tilt 12° / 20°	51
Telescopic Seat Frame for Tilt 30° / Lifter	52
Armrest bracket	53
Legrest Support	54
Seat Plate	55
Telescopic Seat Frame and Sling Seat	56
Telescopic Seat Frame for Tilt 12° / 20°	57
Telescopic Seat Frame for Tilt 30° / Lifter	58
Armrest bracket	53
Legrest Support	59
Sling Seat	60
Telescopic Seat Frame and Universal Adapter	61
Telescopic Seat Frame for Tilt 12° / 20°	51
Telescopic Seat Frame for Tilt 30° / Lifter	52
Armrest bracket	53
Legrest Support	54
Adapter Kit for Universal Seat	62
Seat Cushion	63

Backrest

Backrest	66
Backrest for Standard Seat	67
Backrest Standard Seat, quick release	68
Backrest Standard Seat, settable	69
Backrest Standard Seat, Powered Recliner	70
Backrest for Flex2 Seat	71
Backrest Recliner	72
Backrest Flex2	73
Backrest frame Flex2, tension adjustable	74
Push handles Flex2	75
Push handles Flex2, attendant control	76
Backrest cushion	77
Tension adjustable	78
Backrest for Max Seat	79
Backrest Recliner	80
Backrest Max-Seat	81
Backrest for Modulite Seat	82
Support for Backrest Frame	83
Backrest Frame	84
Sling Back, tension adjustable	85
Backrest support	86
Backposts	87
Connection tube	88
Back Recliner / Stabilization bracket	89
Backrest Cushions	90
Flex 3 / Laguna	91



Yes, you can.®

TABLE OF CONTENTS

Matrx Posture Back	93
Matrx Elite T/R Back	94
Matrx Elite Deep Back	95
Matrx Elite (Full) Back	96

Legrests / Hangers

Legrest Support, 80° & man. elevating - Standard & Modulite Seat	97
Legrest Support, 80° & man. elevating - Flex-2 & Max Seat	98
Legrest Support, Vari series - Standard Seat	99
Legrest Support, Vari series - Flex-2 & Max Seat	100
Legrest Support, Vari series - Modulite Seat	101
Hanger, Std. 80° (01.07.15 =>)	102
Hanger, Std. 80° (=> 30.06.15)	103
Hanger, manually height adjustable	104
Hanger Vari F	105
Hanger Vari A (04/2009 =>)	106
Hanger Vari ADE/ADP electric (07/2005 =>)	107

Footplates / Support tubes

Footrests Vari series	108
Footplate, Standard	109
Footrests 80° & man. elevating	110
Footplate, depth-, angleadjustable, 80°	111
Footrests Vari, Flip-Up (01.08.10 =>)	112
Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)	113
Footboard	114

Armrests

Armrests	115
Armrest for Standard Seat	116
T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>	117
Armrest for Flex2 Seat	118
Armrest long	119
Armrest Hemiplegic	120
Armrest Azalea style	121
T-Armrest detachable	122
Flip-up armrest	123
Mounting bracket Flip-up armrests	124
Skirt guard Flip-up armrest	125
Armrest for Max Seat	126
Armrest long	127
Armrest Hemiplegic	120
T-Armrest detachable	122
Armrests for Modulite Seat	128
T-Armrest	129
Padded Armrest	130
Inrigged Armrest	131
Flip-Up Armrest	132
Following Armrest	133
Hip Support	134



Yes, you can.®

TABLE OF CONTENTS

Castors

Front fork	135
Front fork, suspension	136
Front wheels	137

Drive Wheels

Drive Wheels 14" (8x3.00)	138
---------------------------------	-----

Propulsion

Overview Propulsion SSD	139
-------------------------------	-----

Brakes

Wheel Lock/Parkingbrake	142
-------------------------------	-----

Electronic

Electronic	143
ACS Electronic Modules	144
ACS2 Powermodules	145
ACT for ACS 2	146
ACT-Brackets	147
G-Trac-sensor	148
Lightboard	149
ACS Remote & Brackets	150
REM 24 SD with brackets	151
Remote REM 24SD	152
Remote Bracket Standard	153
Remote Bracket Swing away & height adjust.	154
Remote Bracket Attendant	155
REM A & B with brackets	156
Remote Bracket Standard	153
Remote Bracket Swing away & height adjust.	157
Remote Bracket Attendant	155
REM 550 with brackets	158
Remote Bracket Standard	153
Remote Bracket Swing away & height adjust.	154
Remote Bracket Attendant	155
REM 34 with brackets	159
Remote Bracket Standard	160
Remote Bracket Swing away & height adjust.	161
G90A with brackets	162
Remote Bracket Standard	163
Remote Bracket Swing away & height adjust.	164
Remote Bracket Attendant	165
G91S with brackets	166
Remote Bracket Standard	167
Centre control REM 41	168
Small Chin Control with brackets	169
Small Chincontrol	170
Framework for chincontrol	171



Yes, you can.®

TABLE OF CONTENTS

Dual Control with brackets	172
Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat	173
Remote Bracket Dual Control for Modulite Seat	174
Compact Dual Control with brackets	175
Compact Dual Control	176
Bracket, Compact Dual Control - short	177
Bracket, Compact Dual Control - long	178
Intuitive Dual Control with brackets	179
ACS Cables & Accessories	180
Bus-Cables ACS	181
T-Adapter ACS 2	182
T-Adapter - Modulite Seat	183
Slow-Stop-Sensor & Holder	184
Programming Kits	185
Magnetkey for Remotes	186

Batteries & Chargers

Batteries	187
Battery accessories	188
Battery cable ACS 2	189
Chargers	190

Safety Options & Accessories

Frontlight (German homologation)	191
Rearlight (German homologation)	192
Rear Light Protector Kit	193
LED-Frontlight	194
LED-Rearlight	195
Reflective Sign	196
Kerb Rider	197
Lap Belt	198
Lap Belts (Modulite Seat)	199
Rearview Mirror	200

Seat Accessories

Lateral Support	201
Lateral Support - Flex 2 Seat	202
Lateral Support (Dartex)	203
Lateral Support, extended arm (Dartex)	204
Lateral Support - Modulite Seat	205
Lateral Support (Dartex)	206
Lateral Support, extended arm (Dartex)	204
Headrest	207
Head- / Neckrest	208
Headrest sidewise adjustable	209
Headrest with cheek support	210
Abduction Cushion	211

Accessories

Luggage Rack	212
--------------------	-----



Yes, you can.®

TABLE OF CONTENTS

Mounting Kit Luggage Rack	213
Cane holder (Std., Flex2, Max Seat)	214
Cane holder (Modulite: backrest frame)	215
Cane holder (Modulite: back posts man recliner)	216
Cane holder (Modulite: back posts pow recliner)	217
Fender for Castor	218
Fender for Drive Wheel	219
Trays	220
Hour Counter	222
KLICKfix® bag	223
KLICKfix® bracket for T-Armrest	224
KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite)	225
Toolkit	226
T-Nuts for Accessories Modulite Seat	227

User Manuals Kite

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE SP1548632	User Manual Kite German		1
	CS SP1549795	User Manual Kite czech		1
	EN SP1550136	User Manual Kite English		1
	ES SP1550137	User Manual Kite spanish		1
	FR SP1550138	User Manual Kite French		1
	IT SP1550139	User Manual Kite Italian		1
	NL SP1550140	User Manual Kite dutch		1
	PT SP1550141	User Manual Kite Portuguese		1
	SV SP1550142	User Manual Kite Swedish		1
	PL SP1550208	User Manual Kite polish		1
	HU SP1550209	User Manual Kite hungarian		1
	HR SP1550210	User Manual Kite croatian		1
	SL SP1550211	User Manual Kite Slovenian		1
	FI SP1550344	User Manual Kite finnish		1
	ET SP1552623	User Manual Kite Estonian		1
	LT SP1552624	User Manual Kite Lithuanian		1
	RU SP1555982	User Manual Kite Russian		1

User Manuals REM24SD

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1436142	User Manual REM24SD German		1
	FR 1436143	User Manual REM24SD French		1
	EN 1436144	User Manual REM24SD English		1
	SV 1436145	User Manual REM24SD Swedish		1
	DA 1436146	User Manual REM24SD Danish		1
	NO 1436147	User Manual REM24SD Norwegian		1
	NL 1436148	User Manual REM24SD Dutch		1
	ES 1436149	User Manual REM24SD Spanish		1
	PT 1436150	User Manual REM24SD Portuguese		1
	IT 1436385	User Manual REM24SD Italian		1
	FI 1436386	User Manual REM24SD Finnish		1
	HR SP1533251	User Manual REM24SD Croatian		1
	CS SP1533252	User Manual REM24SD Czech		1
	ET SP1533253	User Manual REM24SD Estonian		1
	HU SP1533254	User Manual REM24SD Hungarian		1
	LT SP1533255	User Manual REM24SD Lithuanian		1
	PL SP1533256	User Manual REM24SD Polish		1
	SL SP1533257	User Manual REM24SD Slovenian		1
	LV SP1543152	User Manual REM24SD Latvian		1

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DA 1529171	User Manual REM A & B Dansk		1
	DE 1529172	User Manual REM A & B German		1
	EN 1529173	User Manual REM A & B English		1
	ES 1529174	User Manual REM A & B Spanish		1
	FI 1529228	User Manual REM A & B Finsk		1
	FR 1529229	User Manual REM A & B French		1
	IT 1529230	User Manual REM A & B Italian		1
	NL 1529231	User Manual REM A & B Dutch		1
	NO 1529232	User Manual REM A & B Norsk		1
	PT 1529233	User Manual REM A & B Portuguese		1
	SV 1529234	User Manual REM A & B Swedish		1
	HR SP1533258	User Manual REM A & B Croatian		1
	CS SP1533259	User Manual REM A & B Czech		1
	ET SP1533265	User Manual REM A & B Estonian		1
	HU SP1533266	User Manual REM A & B Hungarian		1
	LT SP1533267	User Manual REM A & B Lithuanian		1
	PL SP1533268	User Manual REM A & B Polish		1
	SL SP1533269	User Manual REM A & B Slovenian		1
	RO SP1533270	User Manual REM A & B Romanian		1

User Manuals REM550

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DA 1528088	User Manual REM550 Danish		1
	DE 1528089	User Manual REM550 German		1
	EN 1528090	User Manual REM550 English		1
	ES 1528091	User Manual REM550 Spanish		1
	FI 1528092	User Manual REM550 Finsk		1
	FR 1528093	User Manual REM550 French		1
	IT 1528094	User Manual REM550 Italiano		1
	NL 1528095	User Manual REM550 Dutch		1
	NO 1528096	User Manual REM550 Norsk		1
	PT 1528097	User Manual REM550 Portuguese		1
	SV 1528098	User Manual REM550 Swedish		1
	HR SP1533279	User Manual REM550 Croatian		1
	CS SP1533280	User Manual REM551 Czech		1
	ET SP1533281	User Manual REM552 Estonian		1
	HU SP1533282	User Manual REM553 Hungarian		1
	LT SP1533283	User Manual REM554 Lithuanian		1
	PL SP1533284	User Manual REM555 Polish		1
	SL SP1533285	User Manual REM556 Slovenian		1

User Manuals G90

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	1523674	User Manual G90 English		1

User Manuals G91

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1536441	User Manual G91 German		1
	DA 1537929	User Manual G91 Danish		1
	NL 1537930	User Manual G91 Dutch		1
	EN 1537931	User Manual G91 English		1
	FI 1537932	User Manual G91 Finnish		1
	FR 1537933	User Manual G91 French		1
	IT 1537934	User Manual G91 Italian		1
	NO 1537935	User Manual G91 Norwegian		1
	PT 1537936	User Manual G91 Portuguese		1
	ES 1537937	User Manual G91 Spanish		1
	SV 1537938	User Manual G91 Swedish		1

User Manuals REM34 & REM41 Tray/Centre Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1538022	User Manual REM41 Tray Control Danish		1
EN	SP1538023	User Manual REM41 Tray Control English		1
ES	SP1538024	User Manual REM41 Tray Control Spanish		1
FR	SP1538025	User Manual REM41 Tray Control French		1
IT	SP1538026	User Manual REM41 Tray Control Italian		1
NL	SP1538027	User Manual REM41 Tray Control Dutch		1
NO	SP1538028	User Manual REM41 Tray Control Norwegian		1
PT	SP1538029	User Manual REM41 Tray Control Portuguese		1
SV	SP1538030	User Manual REM41 Tray Control Swedish		1
CS	SP1538031	User Manual REM41 Tray Control Czech		1
SL	SP1538032	User Manual REM41 Tray Control Slovenian		1
DE	SP1541402	User Manual REM41 Tray Control German		1

User Manuals IDC Dual Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DE	1521480	User Manual Dual-Control German		1
EN	1521481	User Manual Dual-Control English		1
FR	1521482	User Manual Dual-Control French		1
NL	1521483	User Manual Dual-Control Dutch		1
NO	1521484	User Manual Dual-Control Norwegian		1
SV	1521485	User Manual Dual-Control Swedish		1
DA	1521486	User Manual Dual-Control Danish		1
IT	1522252	User Manual Dual-Control Italian		1
PT	1522254	User Manual Dual-Control Portuguese		1
ES	1522255	User Manual Dual-Control Spanish		1

User Manuals Compact Dual Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
BG	SP1589186	User Manual Compact Dual Control - Bulgarian		1
CS	SP1589313	User Manual Compact Dual Control - Czech		1
DA	SP1588488	User Manual Compact Dual Control - Danish		1
DE	SP1588489	User Manual Compact Dual Control - German		1
EN	SP1588462	User Manual Compact Dual Control - English		1
ES	SP1588496	User Manual Compact Dual Control - Spanish		1
ET	SP1589311	User Manual Compact Dual Control - Estonian		1
FI	SP1588490	User Manual Compact Dual Control - Finnish		1
FR	SP1588491	User Manual Compact Dual Control - French		1
HR	SP1589314	User Manual Compact Dual Control - Croatian		1
HU	SP1589315	User Manual Compact Dual Control - Hungarian		1
IT	SP1588492	User Manual Compact Dual Control - Italian		1
LV	SP1589317	User Manual Compact Dual Control - Latvian		1
LT	SP1589316	User Manual Compact Dual Control - Lithuanian		1
NL	SP1588493	User Manual Compact Dual Control - Dutch		1
NO	SP1588494	User Manual Compact Dual Control - Norwegian		1
PL	SP1589318	User Manual Compact Dual Control - Polish		1
PT	SP1588495	User Manual Compact Dual Control - Portuguese		1
RU	SP1589320	User Manual Compact Dual Control - Russian		1
RO	SP1589319	User Manual Compact Dual Control - Romanian		1
SK	SP1589321	User Manual Compact Dual Control - Slovak		1
SL	SP1589322	User Manual Compact Dual Control - Slovenian		1
SV	SP1588497	User Manual Compact Dual Control - Swedish		1
UK	SP1589323	User Manual Compact Dual Control - Ukrainian		1

User Manuals Headcontrol

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DE	SP1493028	User Manual Headrest Headset German		1
EN	SP1493029	User Manual Headrest Headset English		1
NL	SP1493030	User Manual Headrest Headset Dutch		1

User Manual Proximity Head Remote DSL-105

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1555731	User Manual Proximity Head Remote DSL-105 English		1
DE	SP1555732	User Manual Proximity Head Remote DSL-105 German		1
FR	SP1555733	User Manual Proximity Head Remote DSL-105 French		1
DA	SP1555839	User Manual Proximity Head Remote DSL-105 Danish		1
ES	SP1555840	User Manual Proximity Head Remote DSL-105 Spanish		1
FI	SP1555841	User Manual Proximity Head Remote DSL-105 Finnish		1
IT	SP1555842	User Manual Proximity Head Remote DSL-105 Italian		1
NL	SP1555843	User Manual Proximity Head Remote DSL-105 Dutch		1
NO	SP1555844	User Manual Proximity Head Remote DSL-105 Norwegian		1
PT	SP1555845	User Manual Proximity Head Remote DSL-105 Portuguese		1
SV	SP1555846	User Manual Proximity Head Remote DSL-105 Swedish		1

User Manuals iPortal

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1548231	User Manual iPortal english		1
DA	SP1552493	User Manual iPortal danisch		1
FR	SP1552494	User Manual iPortal french		1
FI	SP1552512	User Manual iPortal finnish		1
NO	SP1552513	User Manual iPortal norwegian		1
SV	SP1552533	User Manual iPortal swedish		1
IT	SP1552534	User Manual iPortal Italian		1
DE	SP1552535	User Manual iPortal german		1
SL	SP1552703	User Manual iPortal slovenian		1
NL	SP1559807	User Manual iPortal Dutch		1
PT	SP1567967	User Manual iPortal Portuguese		1
ES	SP1568026	User Manual iPortal Spanish		1


User Manuals ACS Handprogrammer

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		


Pos.	Part number	Description	Valid from → until	Qty
DE	1419316	User Manual Handheld Programmer German		1
EN	1419317	User Manual Handheld Programmer English		1

Battery box

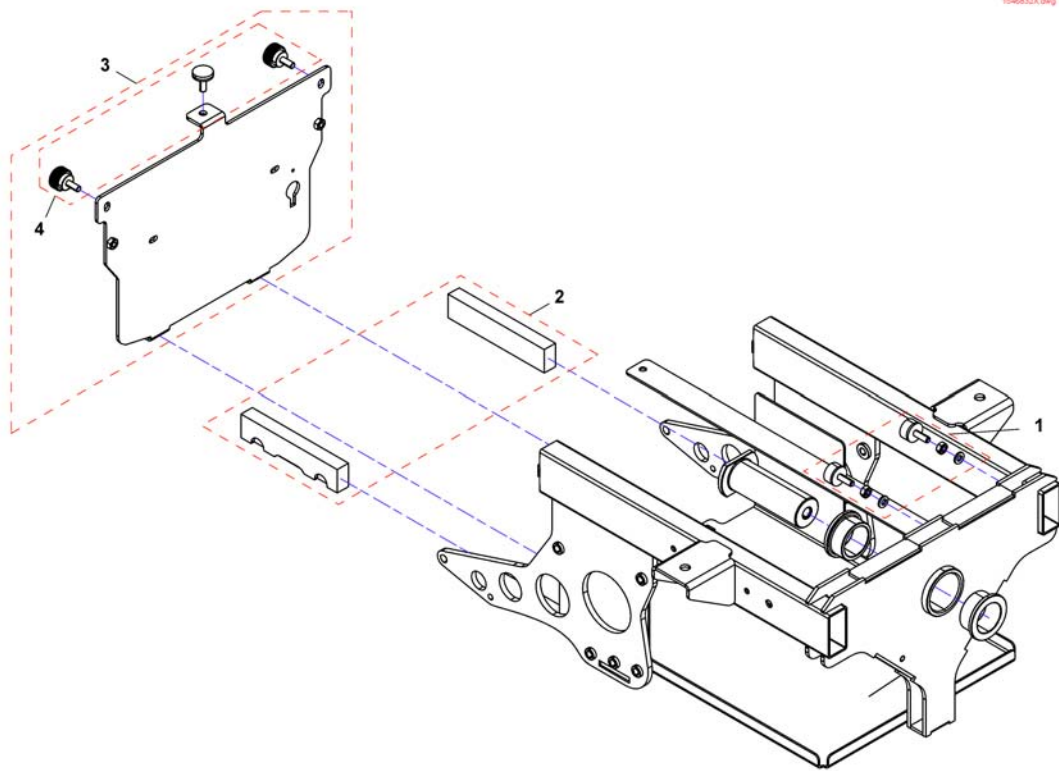


Pos.	Part number	Description	Valid from → until	Qty
1	SP1440366	rubber buffer		1
2	SP1546712	Cable fixation foam		1
3	SP1547993	Electronic support / Chassisflap		1
4	SP1545217	Set of bolts		1

1546834X.dwg

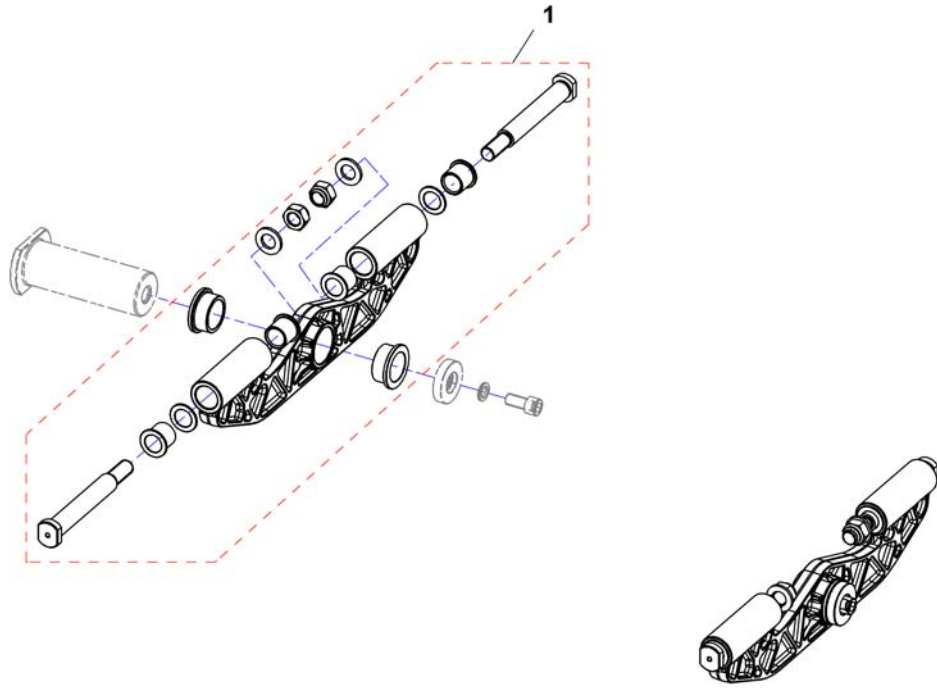

Pos.	Part number	Description	Valid from → until	Qty
1	SP1546834	DST-Bearing		1


Frame

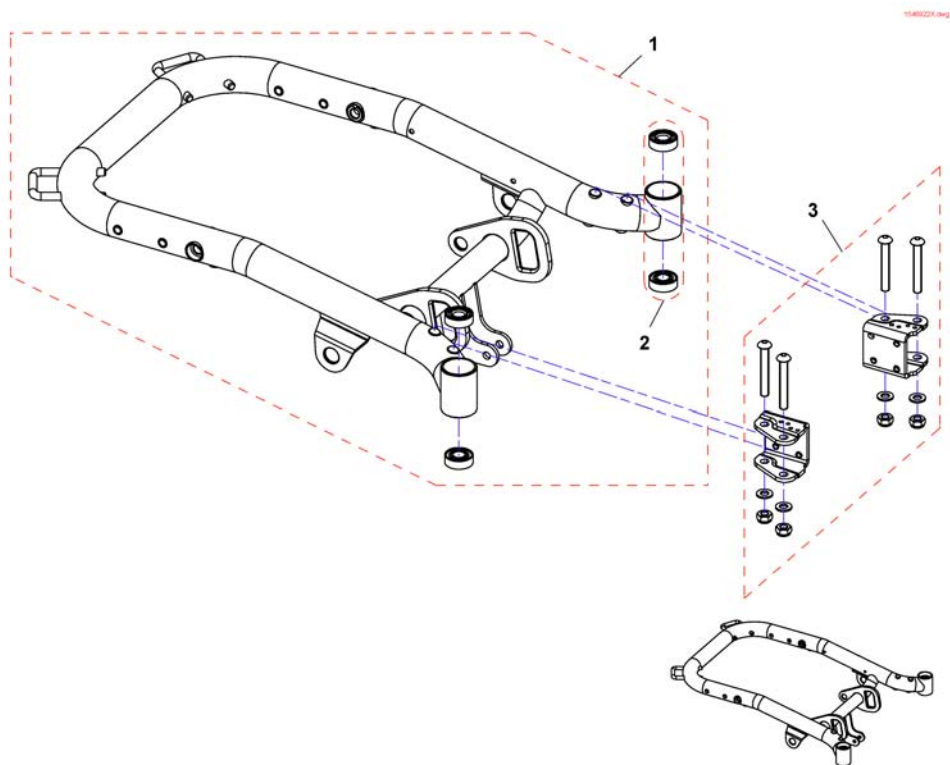

Pos.	Part number	Description	Valid from → until	Qty
-	SP1552601	Touch-up paint, Transparent Blue		1
-	SP1553899	Touch-up paint, Black		1
-	SP1552622	Touch-up paint, Electric Red		1
-	SP1552602	Touch-up paint, Silver Metallic		1
1	SP1546921	Frame cpl. Transparent Blue		1
1	SP1546922	Frame cpl. Black		1
1	SP1546923	Frame cpl. Electric Red		1
1	SP1546924	Frame cpl. Silver Metallic		1
2	SP1528342	Ball Bearing-Kit		1
3	SP1528415	Bearing Plate (pair) <i>only needed for lightoption</i>		1

Suspension


1546918X.dwg

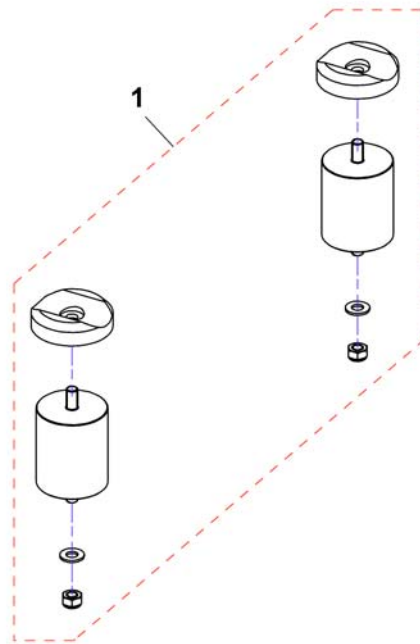

Pos.	Part number	Description	Valid from → until	Qty
1	SP1561822	Suspension soft <90 kg (green dot) <i>up to 90 kg user weight</i>		1
1	SP1546918	Suspension medium (90 - 136 kg) <i>90 to 136 kg user weight</i>		1
1	SP1546919	Suspension hard >136kg (red dot) <i>up to 160 kg user weight</i>		1

Covers


1548233_Ap3

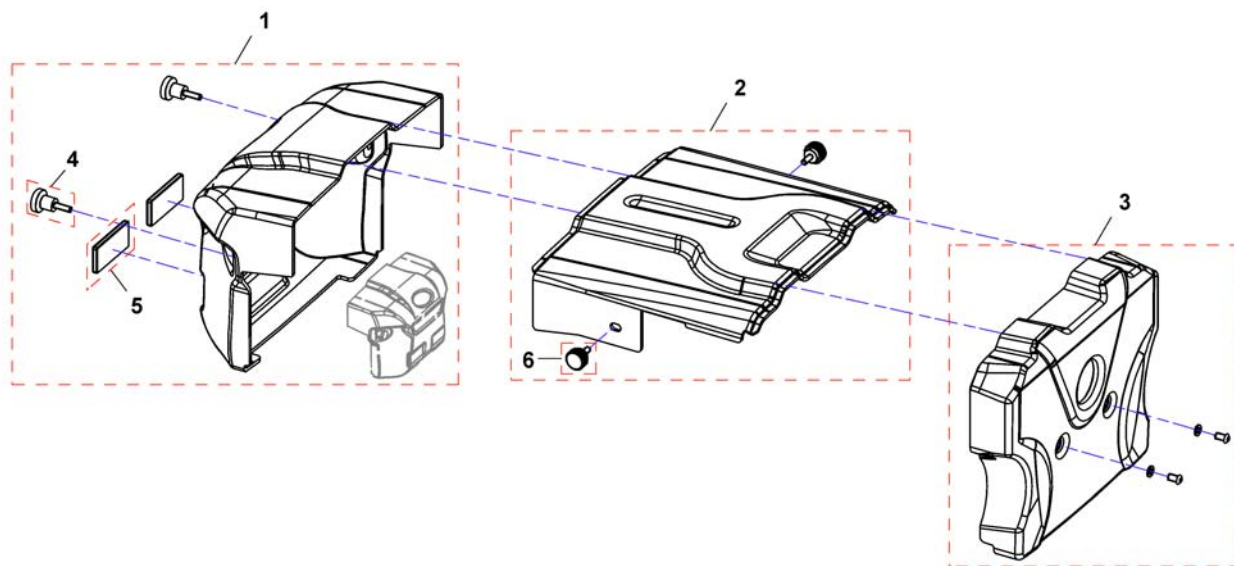

Pos.	Part number	Description	Valid from → until	Qty
1	SP1548894	Rear cover for chairs without light		1
1	SP1550434	Rear cover for chairs with LED-light		1
2	SP1547646	Upper cover		1
3	SP1546833	Front cover		1
4	SP1548240	Knurled screw		1
5	5807542	Reflector angled, red		1
6	1440769	Hand wheel		1

Anti-Tipper


154020X.dwg

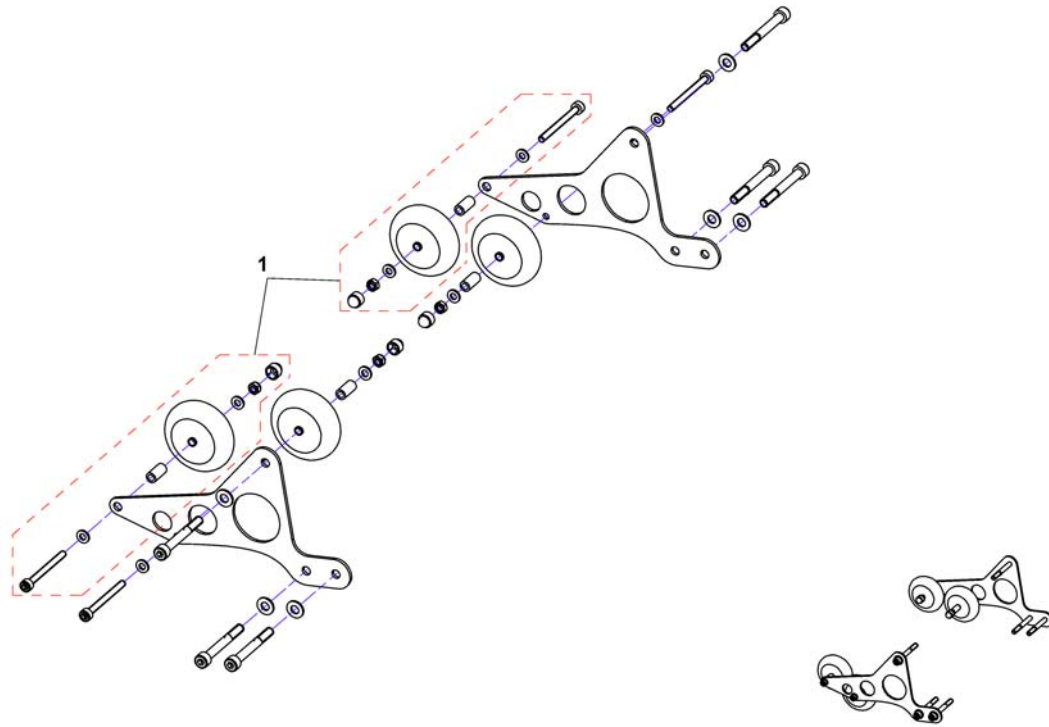

Pos.	Part number	Description	Valid from → until	Qty
1	SP1550694	Anti-Tipp-Wheels Ø70x28		1

Anti-Tipp-Wheels for Lifter

1503723X.dwg

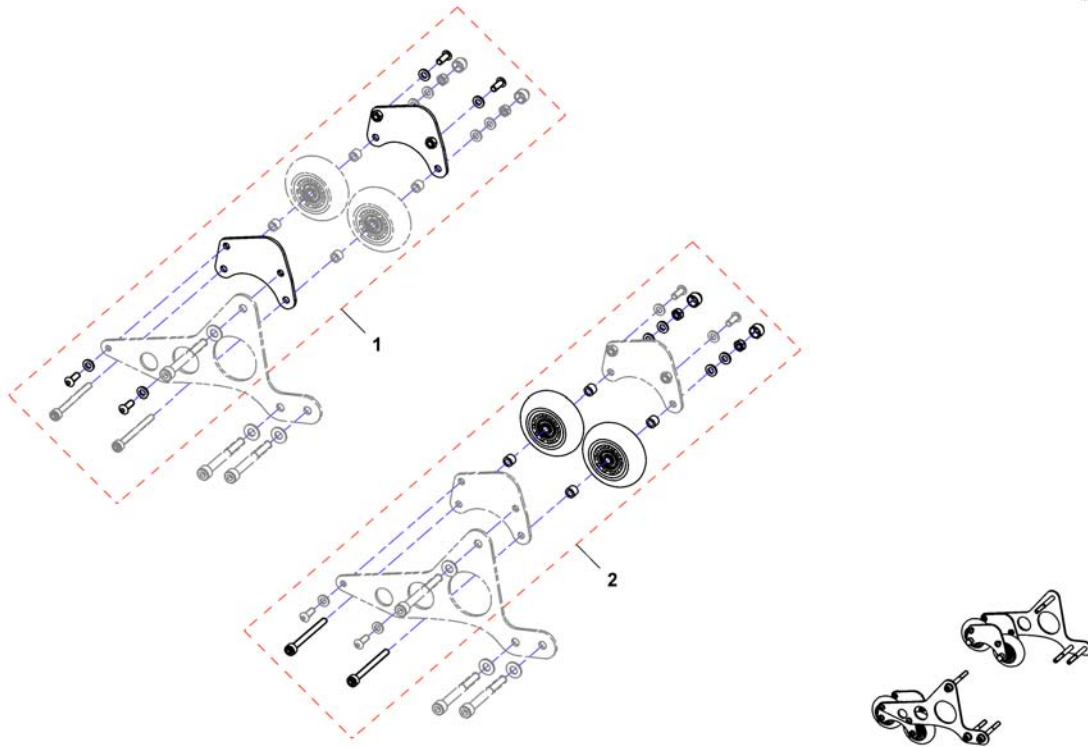

Pos.	Part number	Description	Valid from → until	Qty
1	SP1555072	Brackets Anti-Tipp-Wheels		1
2	SP1555073	Anti-Tipp-Wheels Ø70x28		1

SPOKTE20 160



Pos.	Part number	Description	Valid from → until	Qty
1	-	Standard Seat		1
2	-	Flex2 Seat		1
3	-	Max Seat		1
4	-	Modulite Seat		1


Invacare® Kite | Seat
Standard Seat

1571699.iss



Pos.	Part number	Description	Valid from → until	Qty
1	-	Tilt 12° [1002/1001]		1
2	-	Tilt 12° HD [1002/1001]		1
3	-	Tilt 28° [1014]		1
4	-	Lifter [1021]		1
5	-	Seat Frame & Plate Standard		1
6	-	Seat Cushion [1336/1901]		1


1581881_002

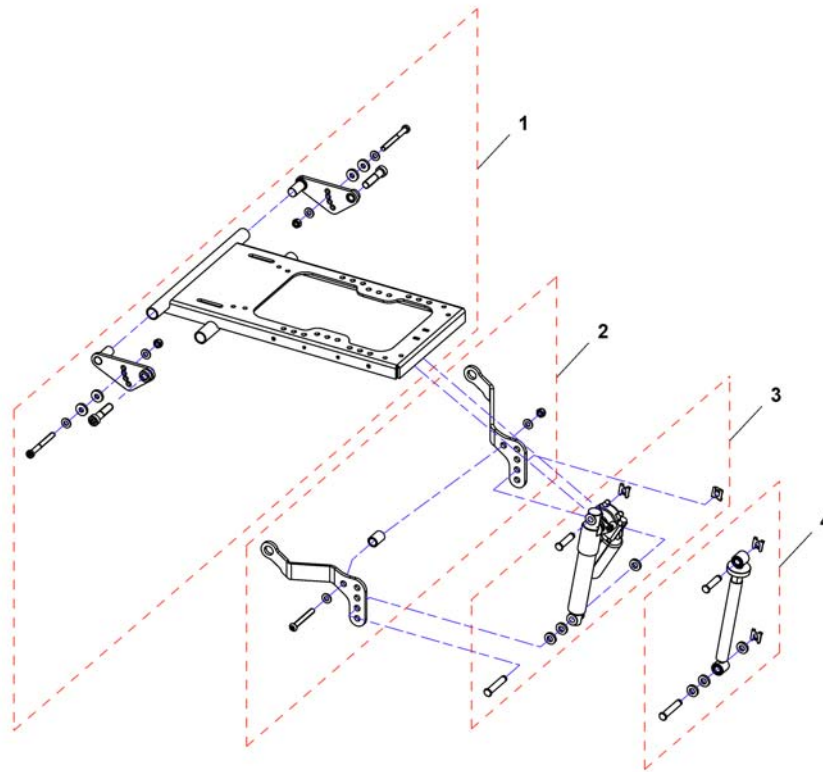


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548532	Seat bracket		1
2	SP1544437	Tilt mounting hardware		1
2	SP1550702	Tilt mounting hardware to adapt kerb climber		1
3	SP1543946	Tilt actuator	→ 2012-09-30	1
3	SP1560776	Tilt actuator	2012-10-01 →	1
3	SP1567927	Tilt actuator <i>For chairs with kerb climber</i>	2012-10-01 →	1
4	SP1548533	Tilt spindle		1
	SP1547305	Extension cable for actuator		1

Tilt 12° HD

1548533.dwg

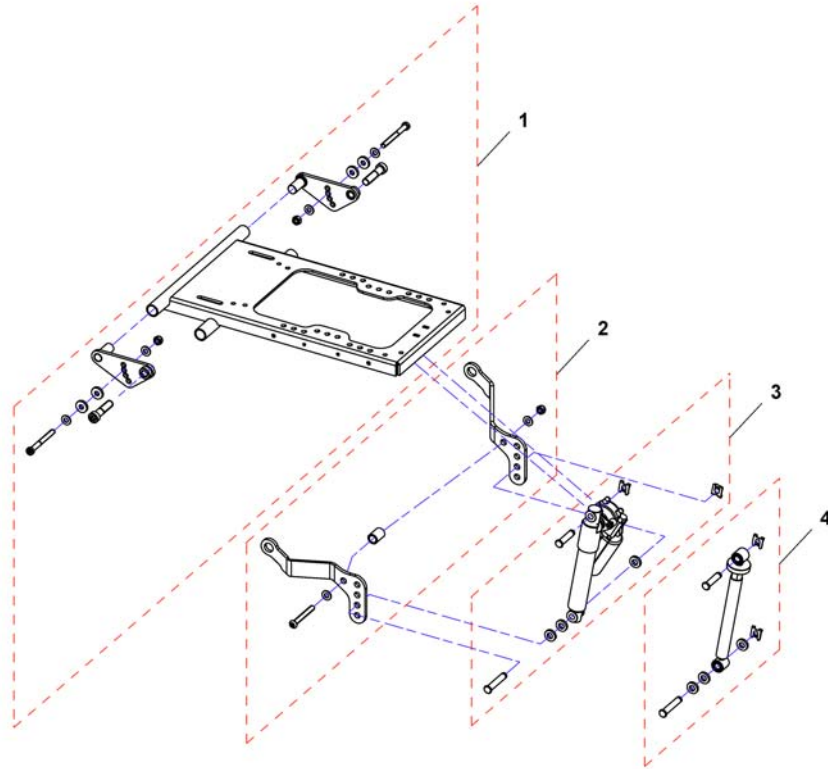


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548532	Seat bracket		1
2	SP1544437	Tilt mounting hardware		1
2	SP1550702	Tilt mounting hardware to adapt kerb climber		1
3	SP1543946	Tilt actuator		1
4	SP1548533	Tilt spindle		1

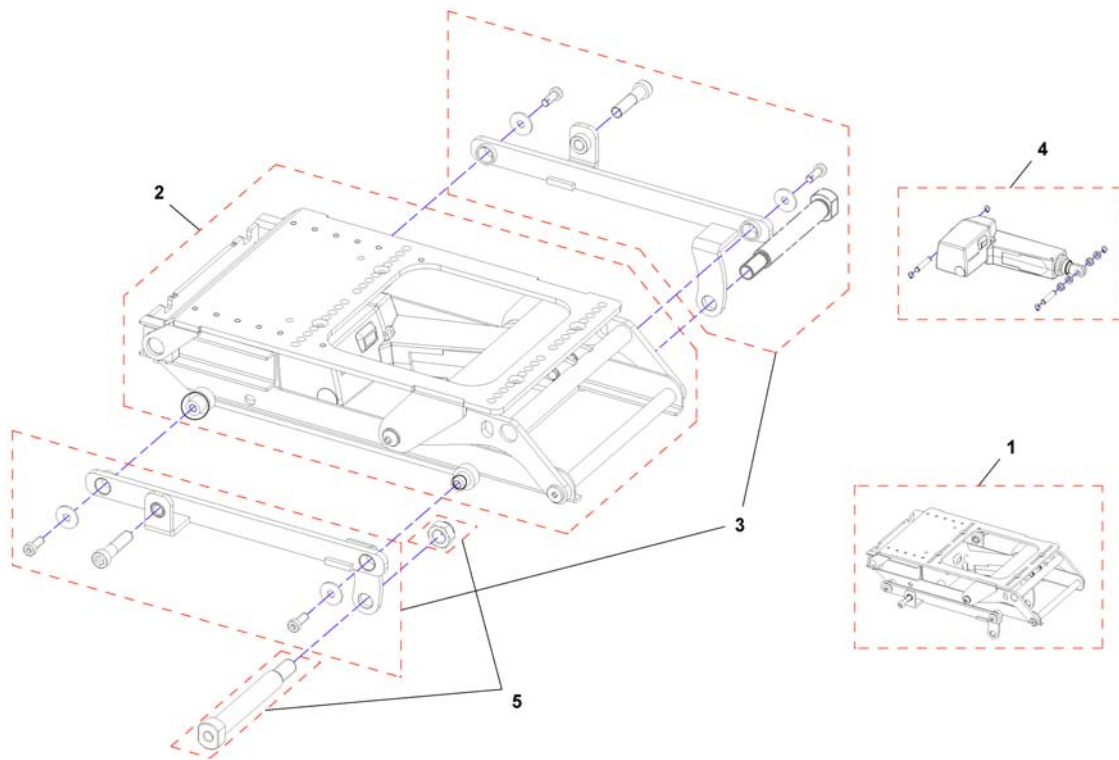

Pos.	Part number	Description	Valid from → until	Qty
1	SP1545645	Seat tilt cpl.		1
2	SP1533063	Tilt unit 0-28°		1
3	SP1550693	Mounting brackets, cpl.		1
4	SP1544888	Actuator		1
5	SP1529400	Bearing bolt kit <i>for Bora only</i>		1

Lifter



1542959X.iso

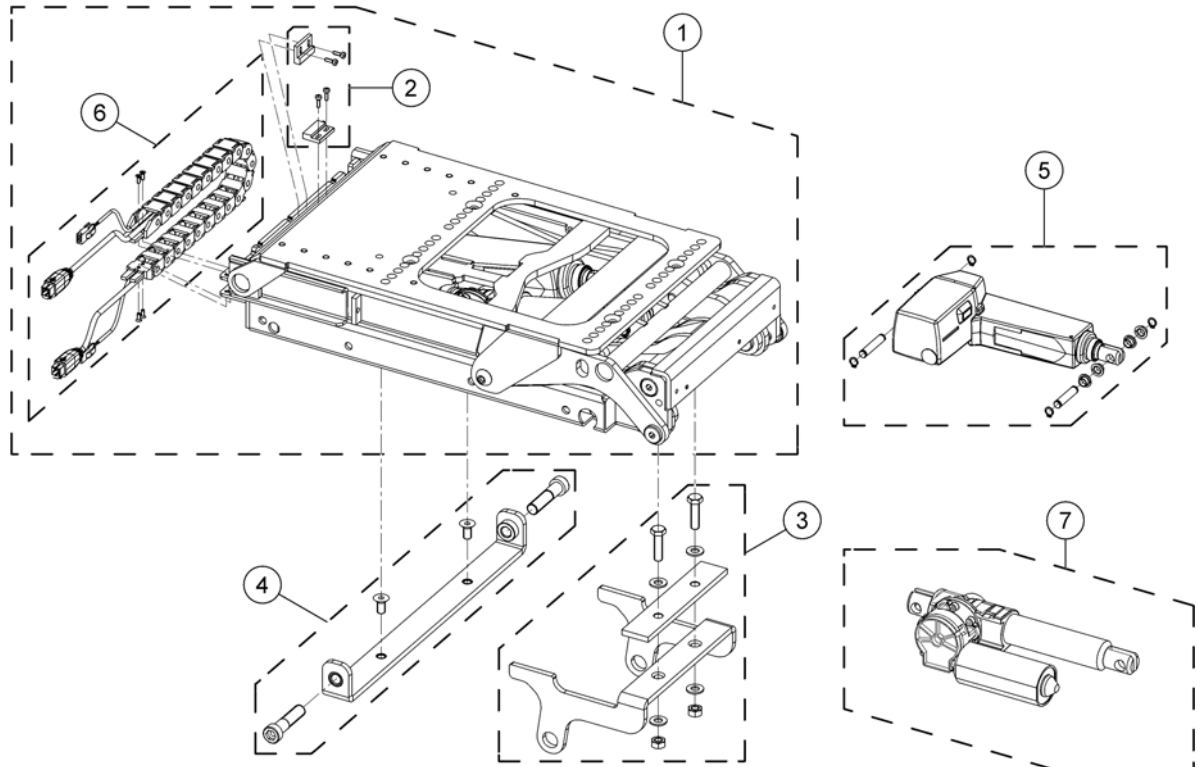

Pos.	Part number	Description	Valid from → until	Qty
1	SP1533423	Lifter		1
2	SP1536184	Speed reduction sensor		1
3	SP1546917	Lifter mounting bracket, front		1
4	SP1546907	Lifter mounting bracket, rear		1
5	SP1533616	Lifter actuator		1
6	SP1552427	Energy chain with cables		1
7	SP1533557	Tilt actuator, ACS2		1
-	SP1571139	Tilt actuator cable ACS2 (~920mm)		1

Seat Frame & Plate Standard


1526884PAC.dwg

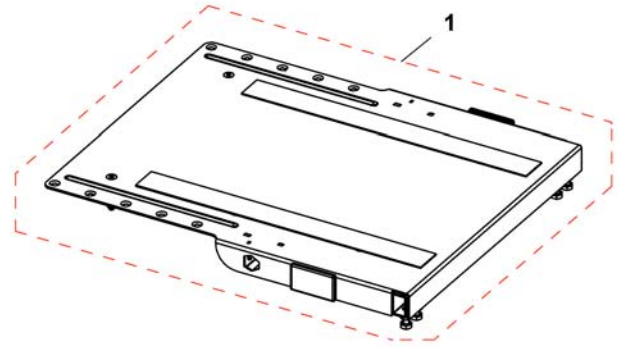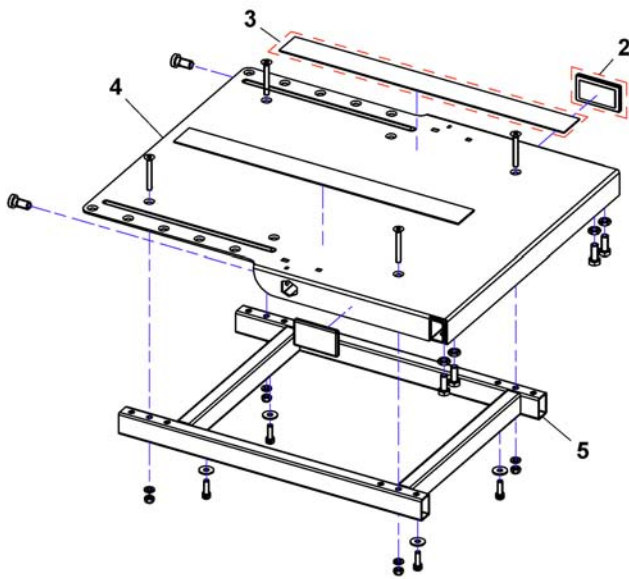

Pos.	Part number	Description	Valid from → until	Qty
1	SP1526884	Seat plate assembly		1
2	P31298	Reflector, yellow		1
3	SPU21087	Velcro		1
4	1419999-9005	Seat Plate		1
5	SP1523467	Seat frame		1

Seat Cushion


1571799.00

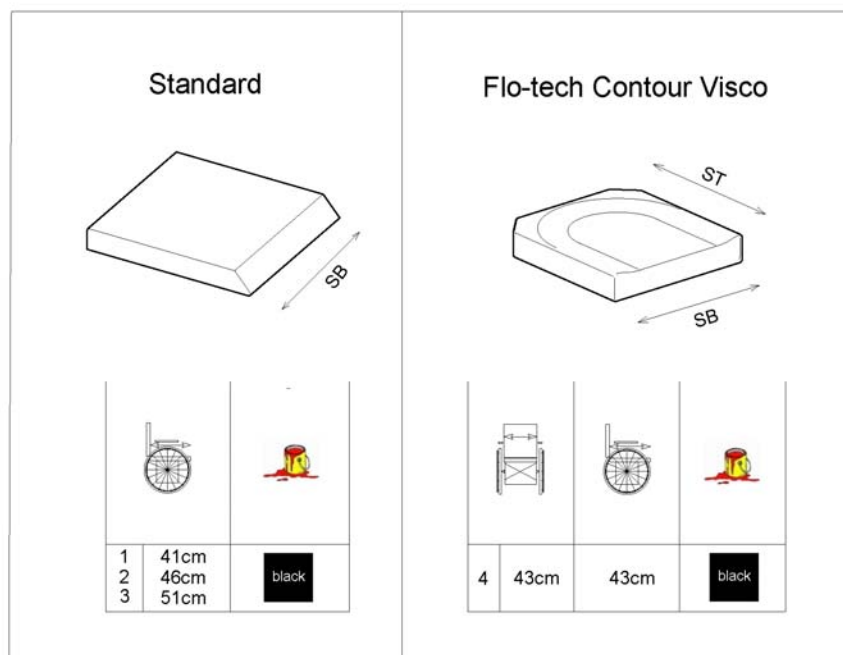

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard SD 410mm [1336+0326]		1
2	1420461	Seat Cushion Standard SD 460mm [1336+0327]		1
3	1420462	Seat Cushion Standard SD 510mm [1336+0330]		1
4	1501113	Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm [1901]		1


Flex2 Seat

1571699.iss

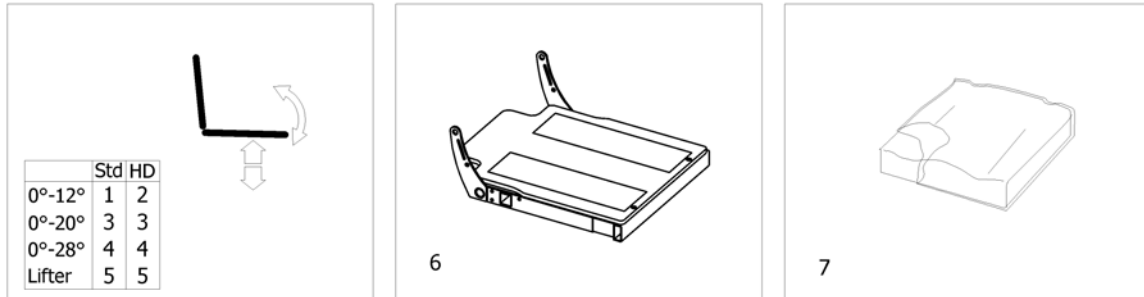

Pos.	Part number	Description	Valid from → until	Qty
1	-	Tilt 12° [1002/1001]		1
2	-	Tilt 12° HD [1002/1001]		1
3	-	Tilt 20° [1045]		1
4	-	Tilt 28° [1014]		1
5	-	Lifter [1021]		1
6	-	Seat Frame with Plate		1
7	-	Seat cushion		1

Tilt 20°

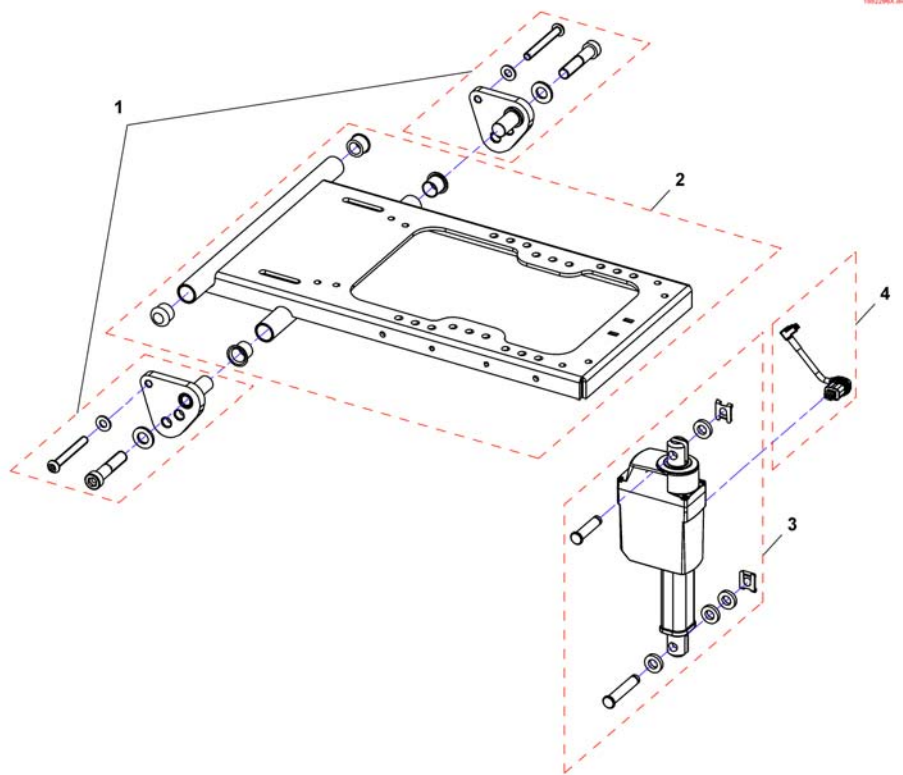


1552968.dwg

Pos.	Part number	Description	Valid from → until	Qty
1	SP1553666	Tilt adjustment bracket		1
2	SP1553667	Seat bracket		1
3	SP1553668	Actuator LA23, cpl.		1
4	SP1553669	Actuator cable		1

Seat Frame with Plate


1531262k.dwg

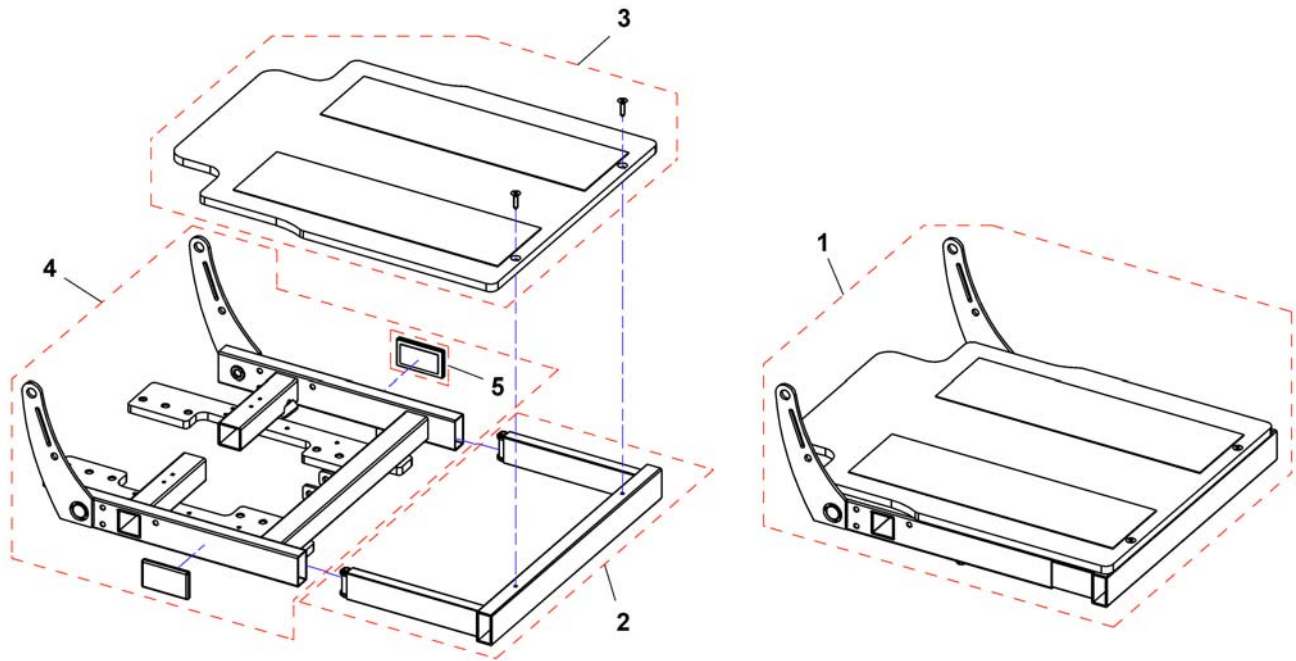


Pos.	Part number	Description	Valid from → until	Qty
1	SP1531262	Flex2 Seat frame for SW 39		1
1	SP1531263	Flex2 Seat frame for SW 44		1
1	SP1531264	Flex2 Seat frame for SW 49		1
2	SP1528416	Legrest attachment SW 39		1
2	SP1528417	Legrest attachment SW 44		1
2	SP1528418	Legrest attachment SW 49		1
3	SP1528419	Seat plate SW 39		1
3	SP1528420	Seat plate SW 44		1
3	SP1528421	Seat plate SW 49		1
4	SP1536658	Seat frame SW 39		1
4	SP1536659	Seat frame SW 44		1
4	SP1536660	Seat frame SW 49		1
5	P31298	Reflector, yellow		1

Seat cushion

1511708 (en)

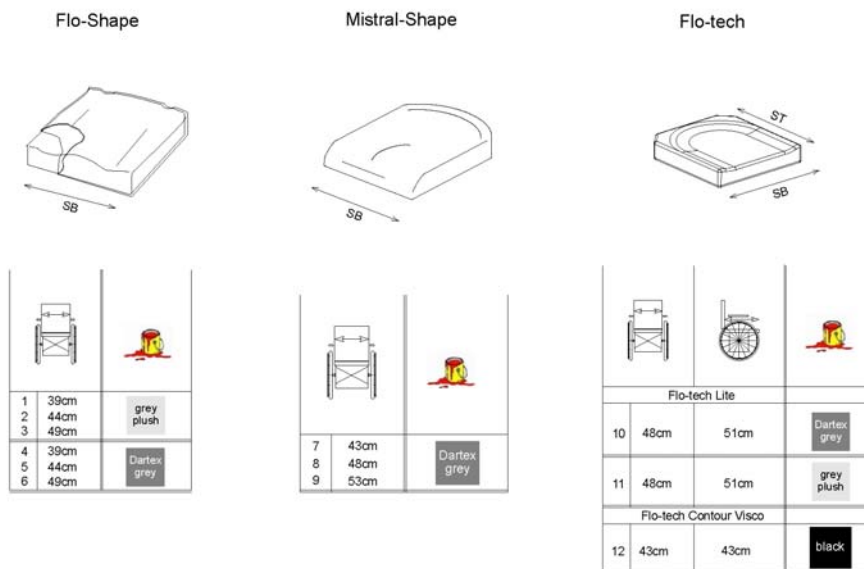


Pos.	Part number	Description	Valid from → until	Qty
1	1508054	Flo-Shape cushion SW 355 x SD 535 Grey plush [1464+0289]		1
2	1508056	Flo-Shape cushion SW 405 x SD 535 Grey plush [1465+0289]		1
3	1508058	Flo-Shape cushion SW 455 x SD 535 Grey plush [1466+0289]		1
4	1508053	Flo-Shape cushion SW 355 x SD 535 Dartex [1464+0278]		1
5	1508055	Flo-Shape cushion SW 405 x SD 535 Dartex [1465+0278]		1
6	1508057	Flo-Shape cushion SW 455 x SD 535 Dartex [1466+0278]		1
7	1435856	SEAT CUSHION 365X490, MISTRAL2, DARTEX GREY [1470]		1
8	1435857	SEAT CUSHION 415X490, MISTRAL2, DARTEX GREY [1321]		1
9	1435858	SEAT CUSHION 465X490, MISTRAL2, DARTEX GREY [1471]		1
10	SP1525259-0232	Easy-Adapt cushion (Flo-tech Lite) SW 48 x SD 51cm Dartex [1978+0278]		1
11	SP1525259-0237	Easy-Adapt cushion (Flo-tech Lite) SW 48 x SD 51cm Grey Plush [1978+0289]		1
12	1501113	Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm [1901]		1

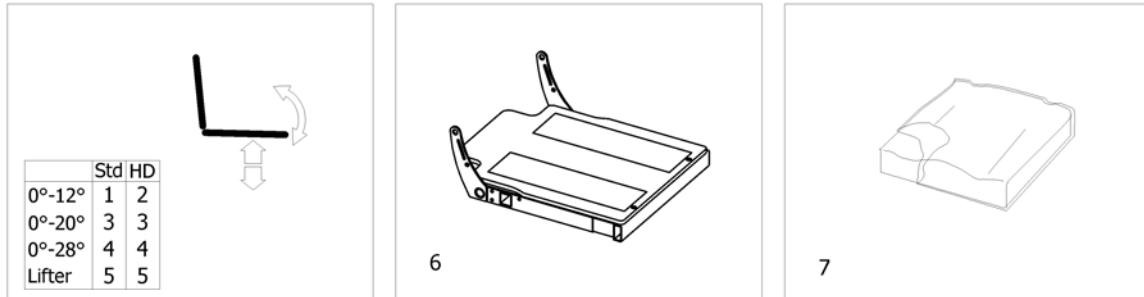

Pos.	Part number	Description	Valid from → until	Qty
1	-	Tilt 12° [1002/1001]		1
2	-	Tilt 12° HD [1002/1001]		1
3	-	Tilt 20° [1045]		1
4	-	Tilt 28° [1014]		1
5	-	Lifter [1021]		1
6	-	Seat Frame with Plates		1
7	-	Seat Cushion Max [1459]		1


Seat Frame with Plates

15312826.dwg

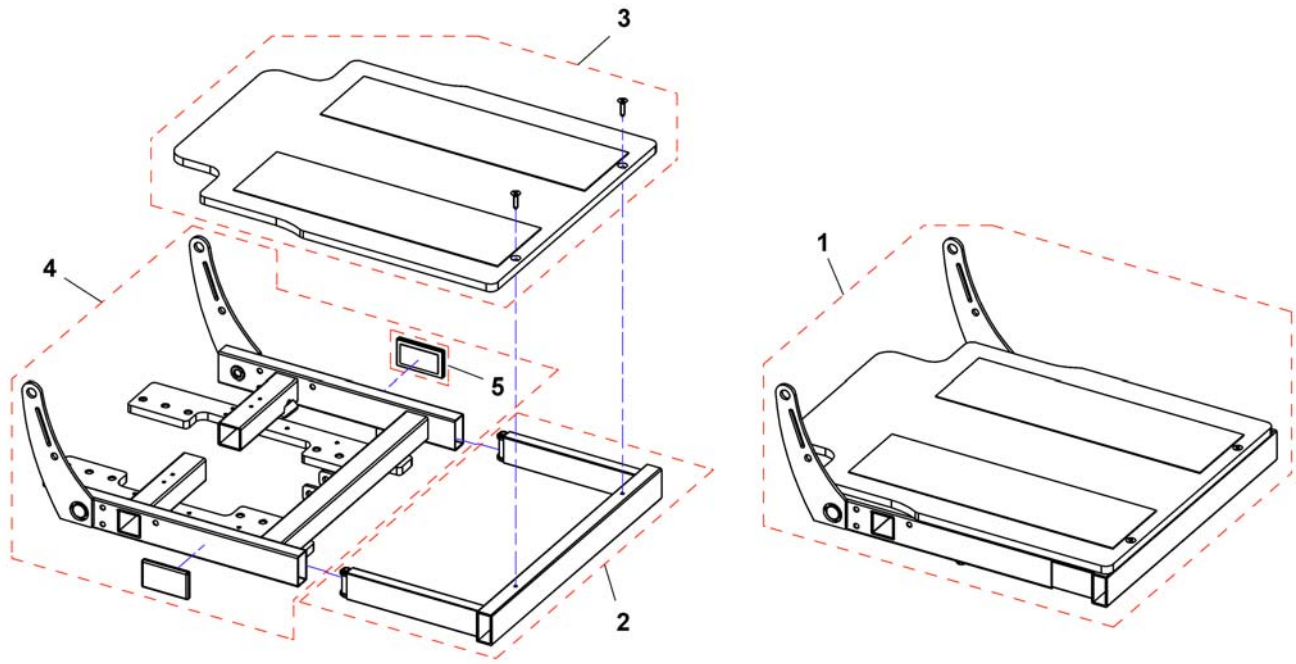


Pos.	Part number	Description	Valid from → until	Qty
1	SP1551683	Seat frame for 5		1
2	SP1523723	Legrest attachment SW 55		1
3	SP1525083	Seat plate SW 550mm		1
4	SP1544672	Seat frame SW 55		1
5	P31298	Reflector, yellow		1


1511723 (en)

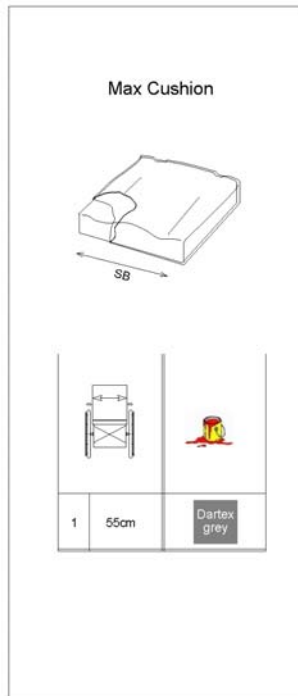

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536595	Seat cushion 550 x 635, TR23 [1459]		1

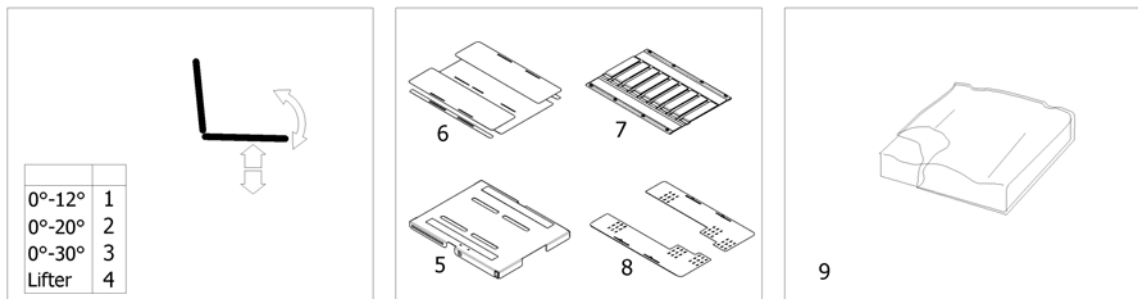


Pos.	Part number	Description	Valid from → until	Qty
1	-	Tilt 12° [1091]		1
2	-	Tilt 20° [1092]		1
3	-	Tilt 30° [1093]		1
4	-	Lifter [1094]		1
5	-	One Piece Seat Plate [2110]		1
6	-	Telescopic Seat Frame and Plate [2111]		1
7	-	Telescopic Seat Frame and Sling Seat [2112]		1
8	-	Telescopic Seat Frame and Universal Adapter [2109]		1
9	-	Seat Cushion		1

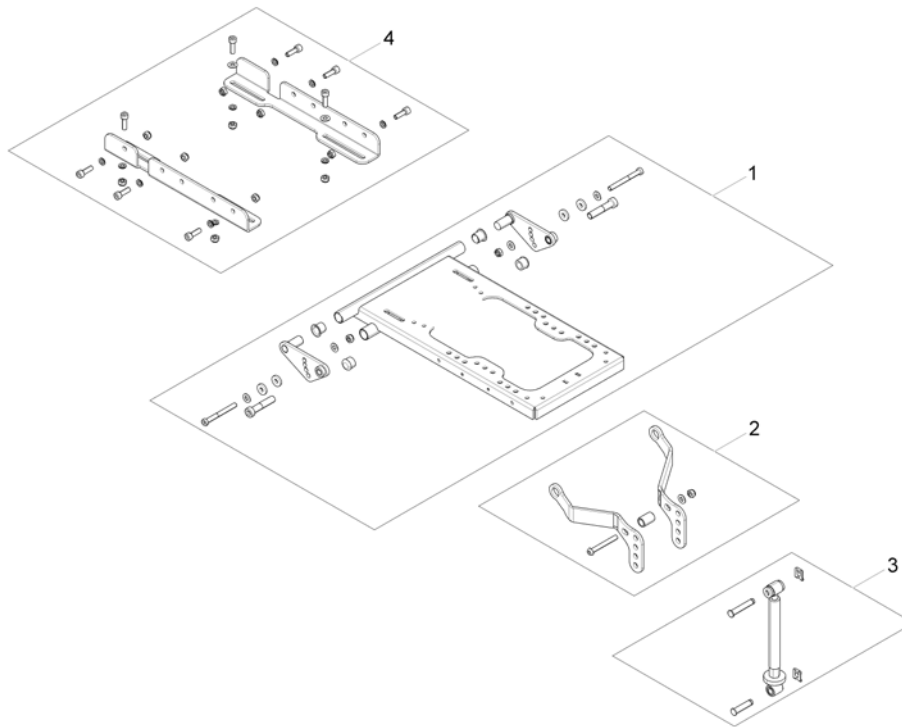

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571990	Seat bracket		1
2	SP1544437	Tilt mounting hardware		1
2	SP1550702	Tilt mounting hardware to adapt kerb climber		1
3	SP1548533	Tilt spindle		1
4	SP1563231	Mounting Rails		1


Tilt 20°

1572198.00

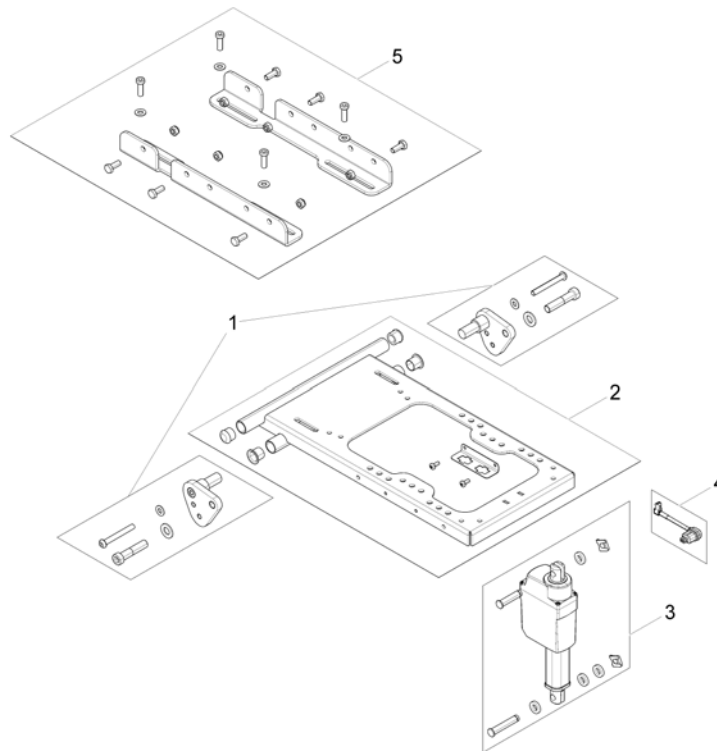

Pos.	Part number	Description	Valid from → until	Qty
1	SP1553666	Tilt adjustment bracket		1
2	SP1553667	Seat bracket		1
3	SP1585226	Actuator 'fast' Tilt 20° Kite, cpl.		1
4	SP1553669	Actuator cable		1
5	SP1563231	Mounting Rails		1


1571872.00

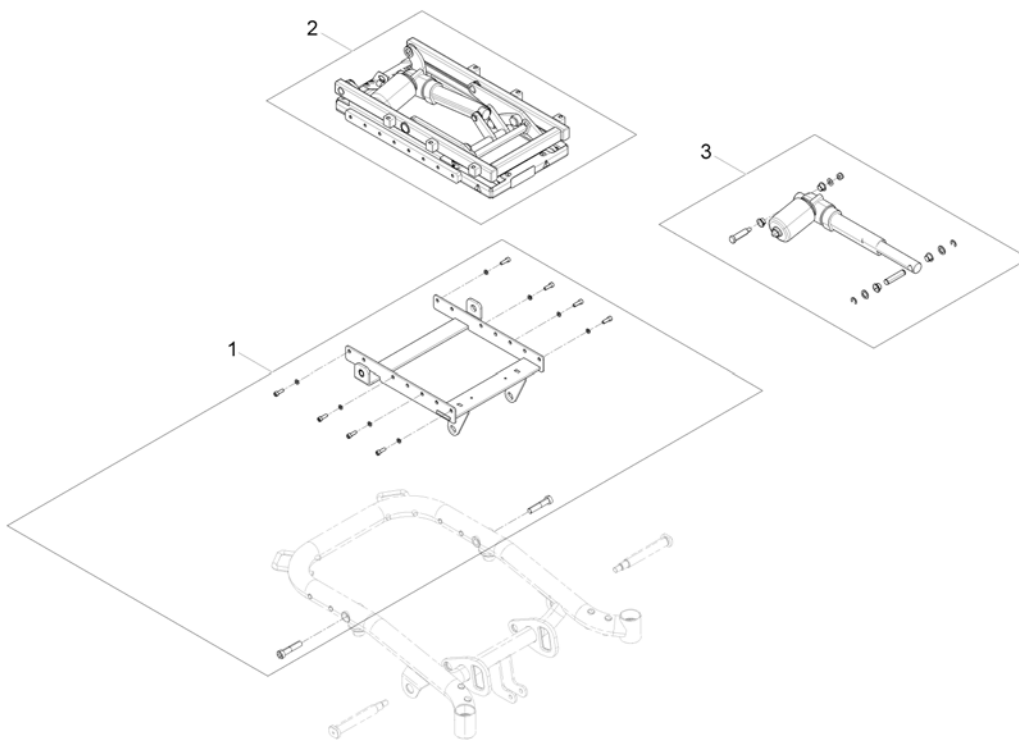

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571705	Lifter- / Tilt bracket		1
2	SP1558714	Tilt unit		1
3	SP1571809	Actuator		1
-	SP1577616	Adapter Cable		1

Lifter


1572885.iso

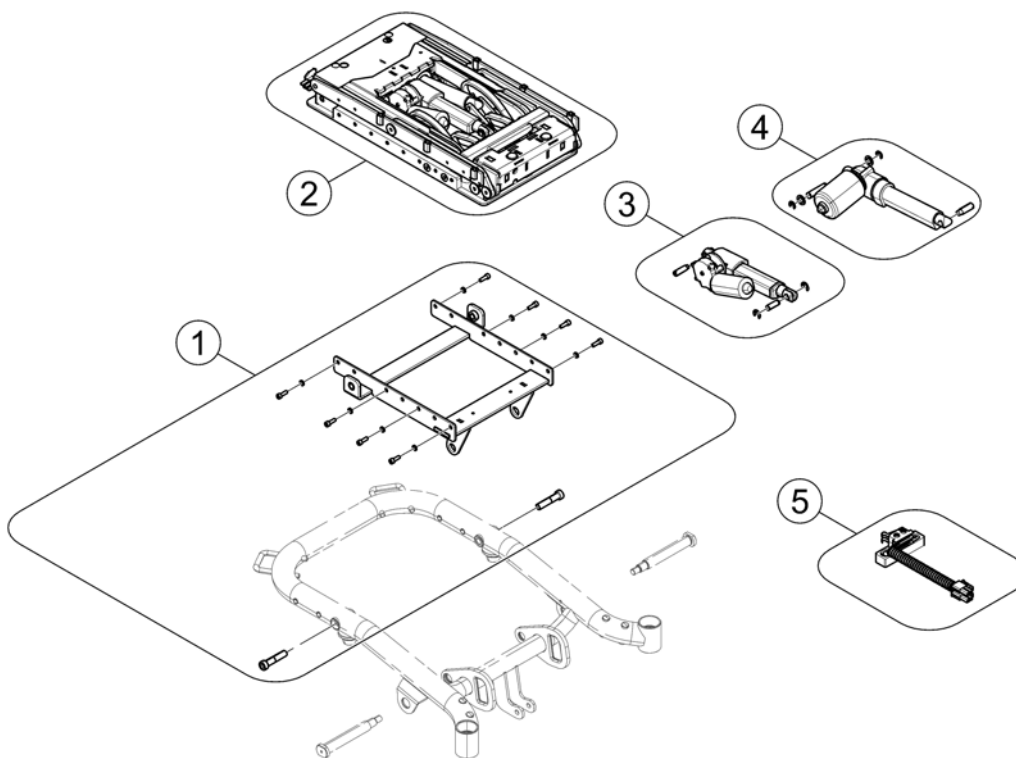

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571705	Lifter- / Tilt bracket		1
2	SP1558715	Lifter- / Tilt unit		1
3	SP1571822	Tilt actuator		1
4	SP1571823	Lifter actuator		1
5	SP1577762	Slow-Stop Switchcable with Mounting		1
-	SP1570291	Adapter Cable		1
-	SP1570294	Extension Cable		1
-	SP1570295	Extension Cable B		1

One Piece Seat Plate


1571064.000

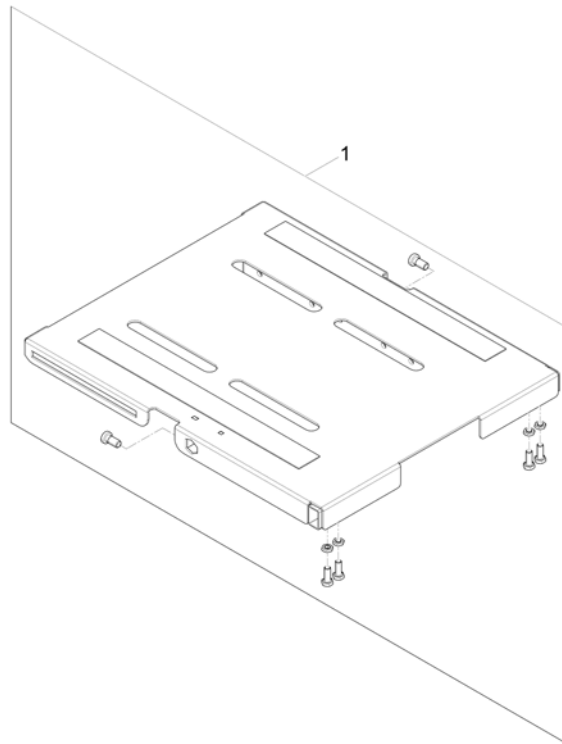

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571297	One Piece Seat Plate		1

Telescopic Seat Frame and Plate


1572312 iso

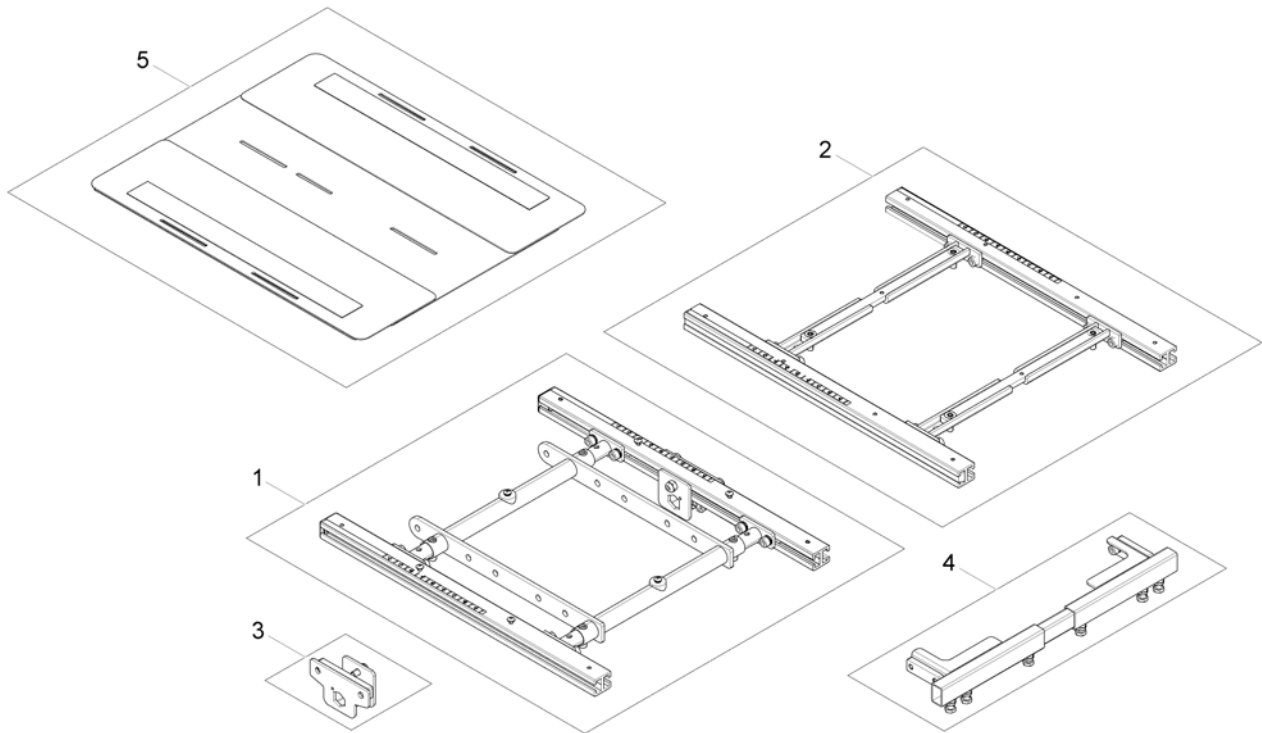

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 12° / 20°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Seat Plate		1

Telescopic Seat Frame for Tilt 12° / 20°


1572009.iss

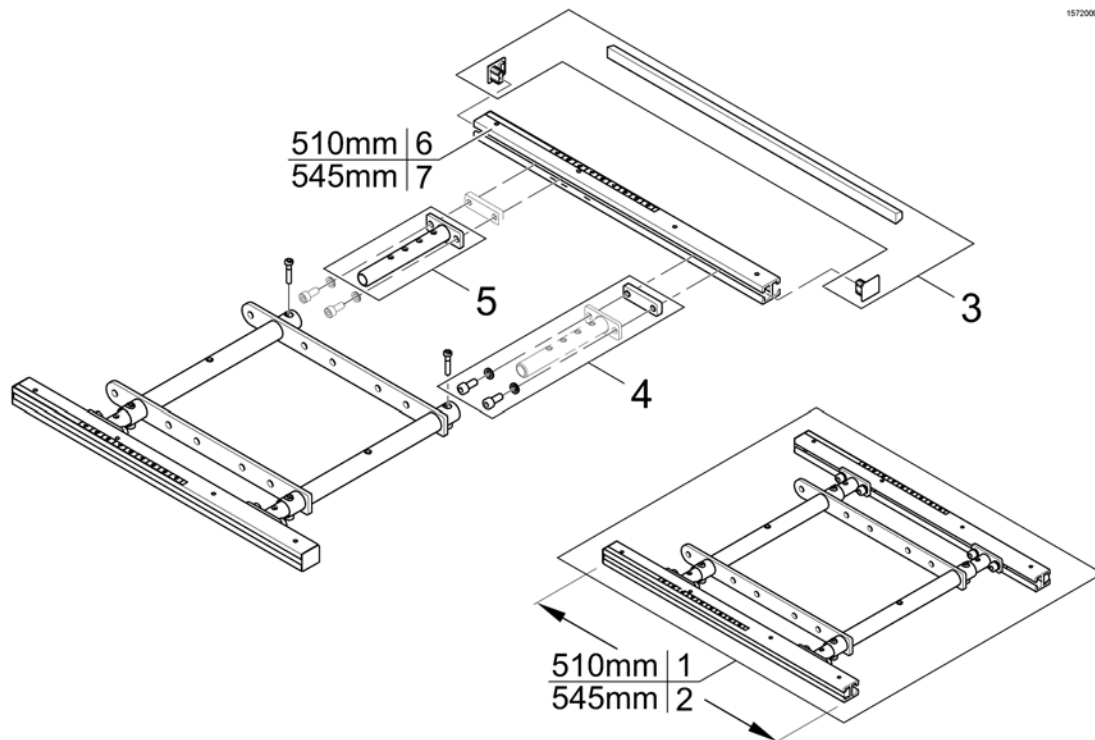

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572977	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582216	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1554646	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1


Telescopic Seat Frame for Tilt 30° / Lifter

1572004 Rev

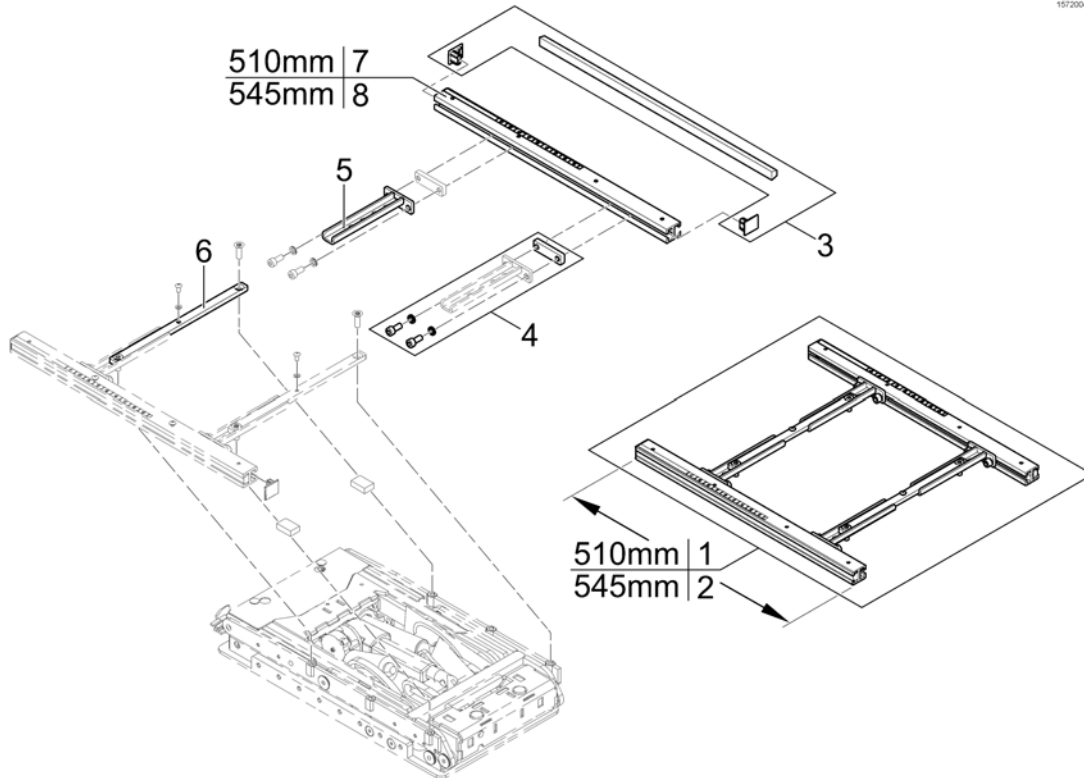

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572992	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582219	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1567007	Seat Frame Adapter		1
6	SP1567008	Middle Rail		2
7	SP1588443	Seat Rails 510mm (2pcs kit)		1
8	SP1588444	Seat Rails 545mm (2pcs kit)		1


1571834 Rev

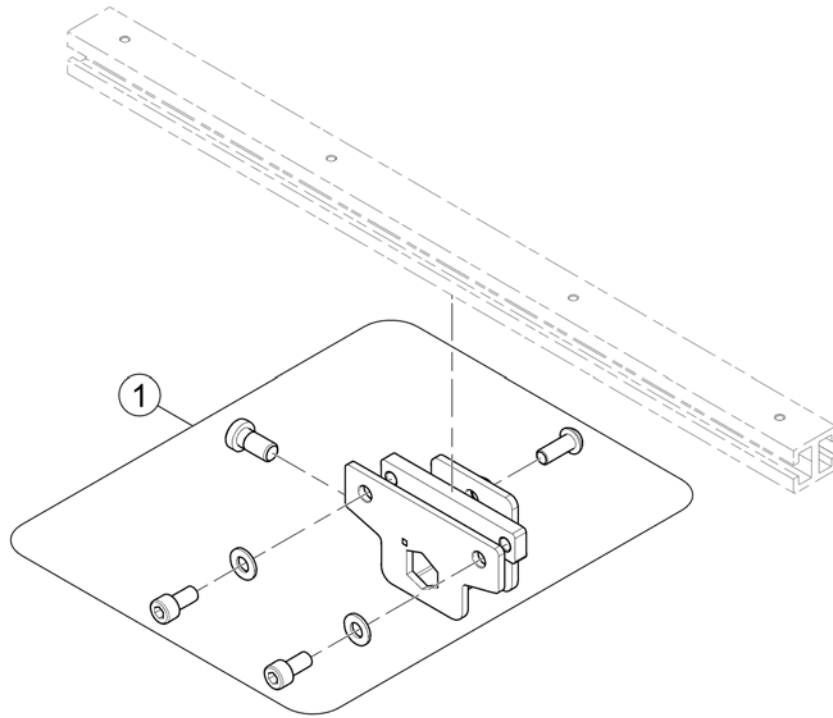


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570189	Armrest bracket		1

Legrest Support

1572021.000

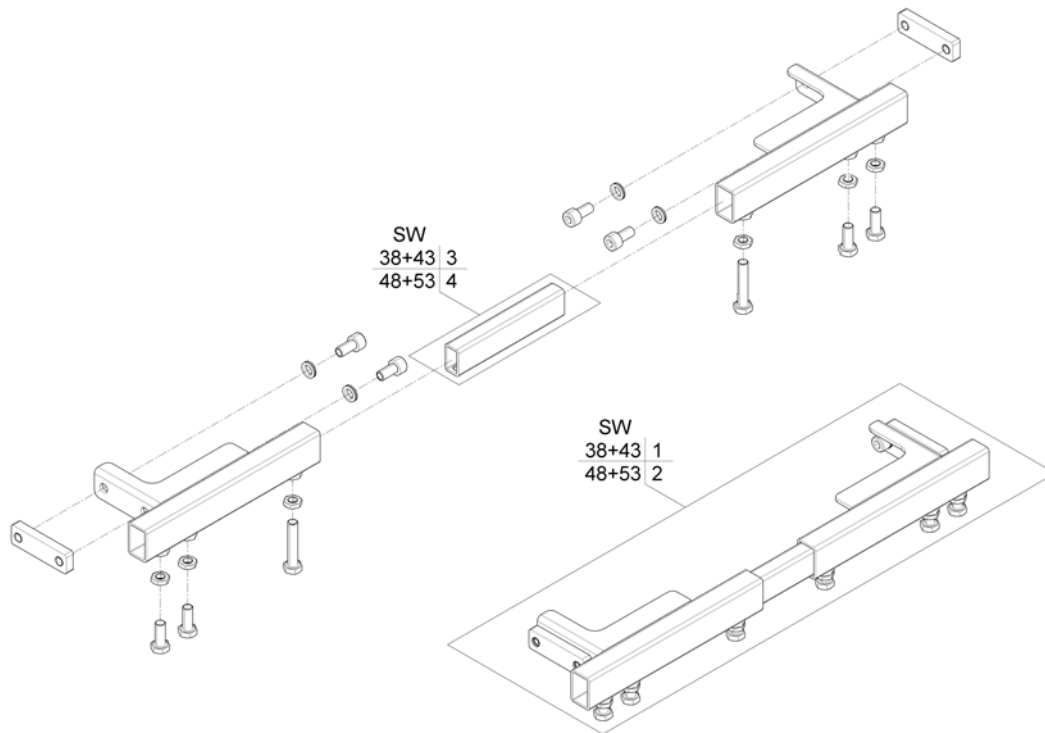


Pos.	Part number	Description	Valid from → until	Qty
1	SP1558935	Legrest Support SW38+43, cpl.		1
2	SP1559294	Legrest Support SW48+53, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

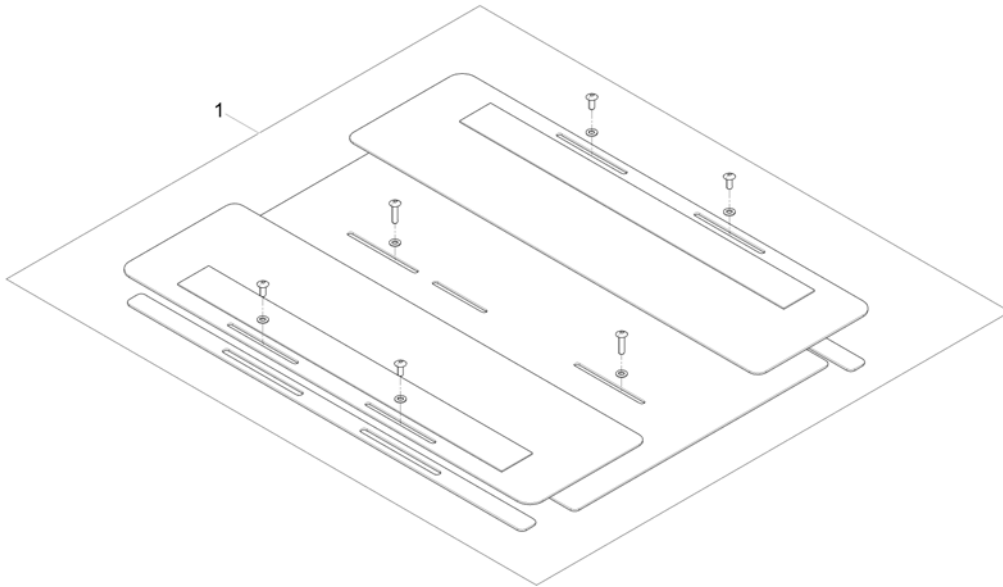

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572199	Seat Plate 510mm, cpl.		1

Telescopic Seat Frame and Sling Seat


1572311.00

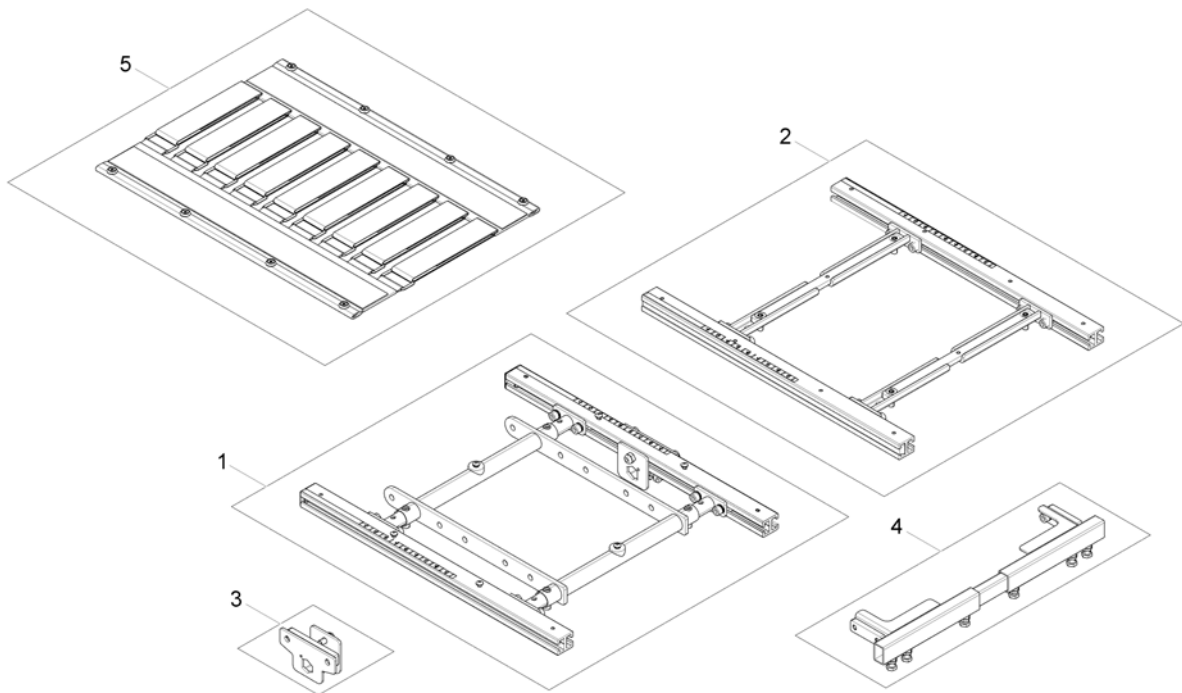

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 12° / 20°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Sling Seat		1

Telescopic Seat Frame for Tilt 12° / 20°


1571950.kim

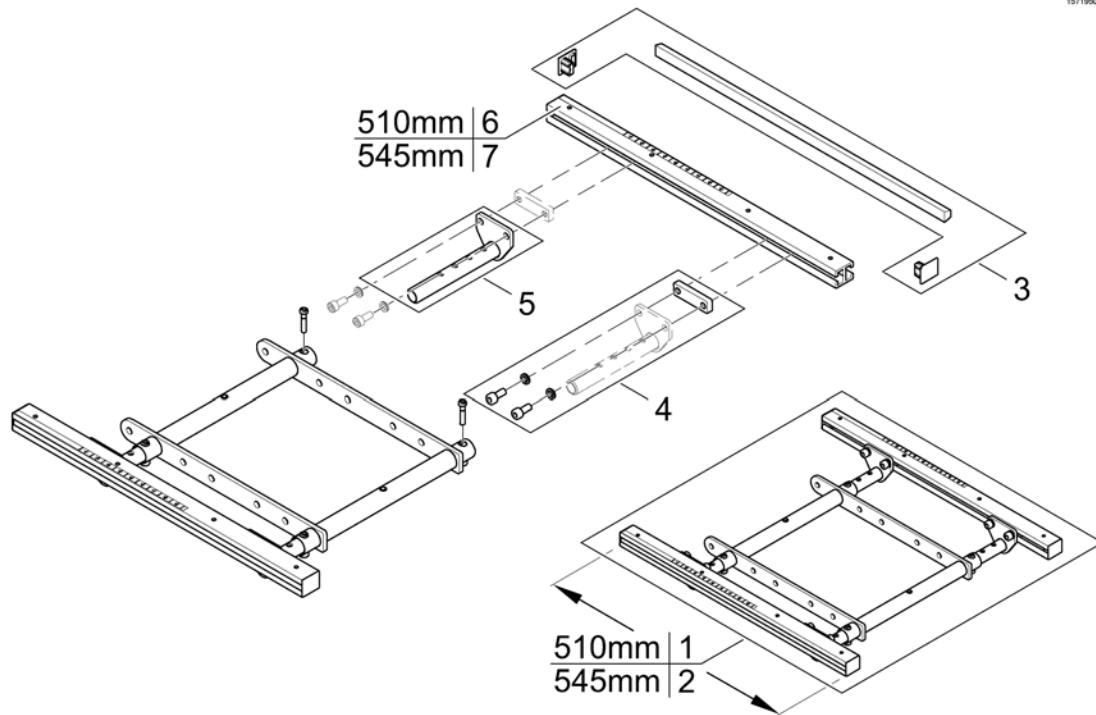


Pos.	Part number	Description	Valid from → until	Qty
1	SP1572978	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582217	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1555400	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1

Telescopic Seat Frame for Tilt 30° / Lifter

1571832 No

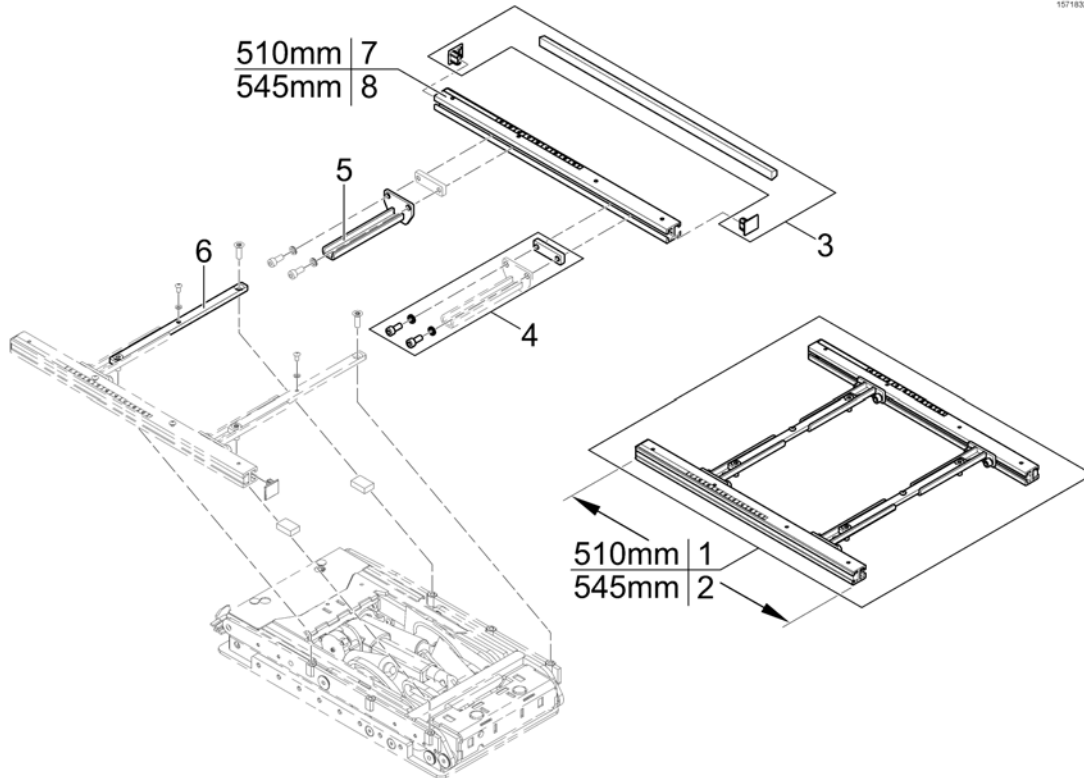


Pos.	Part number	Description	Valid from → until	Qty
1	SP1572993	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582230	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1567720	Seatframe Adapter		1
6	SP1567008	Middle Rail		1
7	SP1588443	Seat Rails 510mm (2pcs kit)		1
8	SP1588444	Seat Rails 545mm (2pcs kit)		1

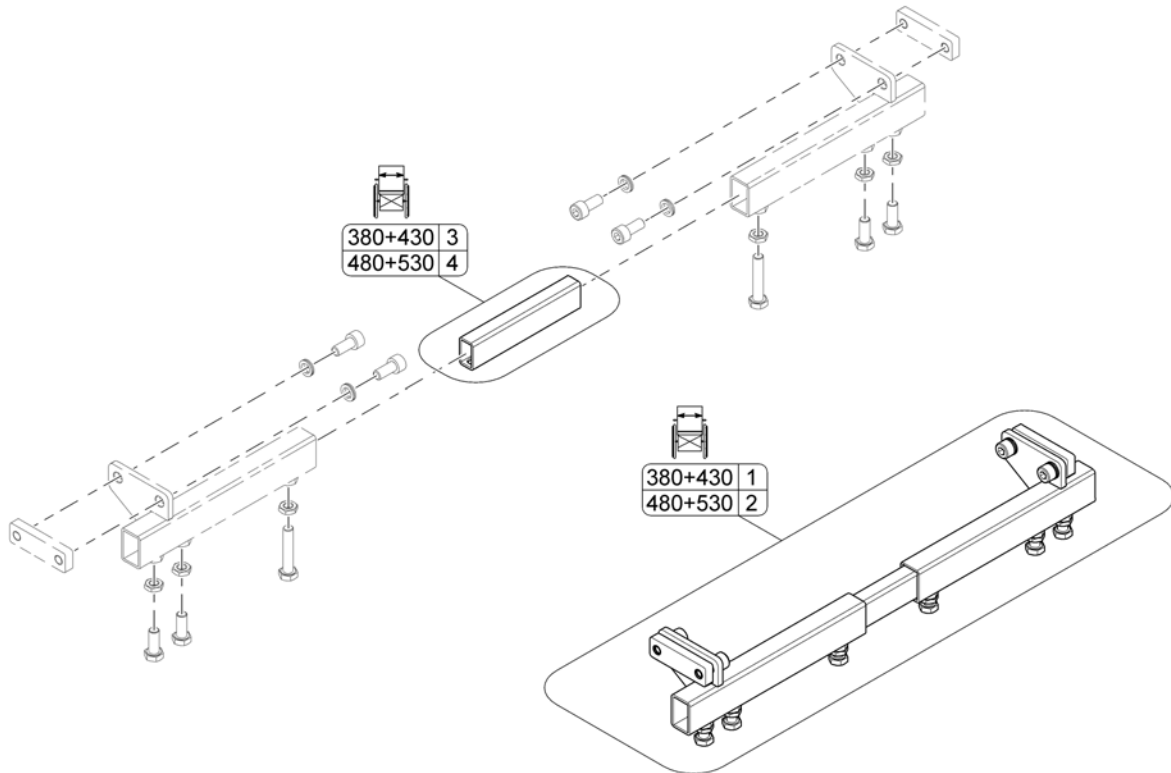

Pos.	Part number	Description	Valid from → until	Qty
1	SP1559305	Legrest Support SW380+430mm, cpl.		1
2	SP1559307	Legrest Support SW480+530mm, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

Sling Seat

151176290

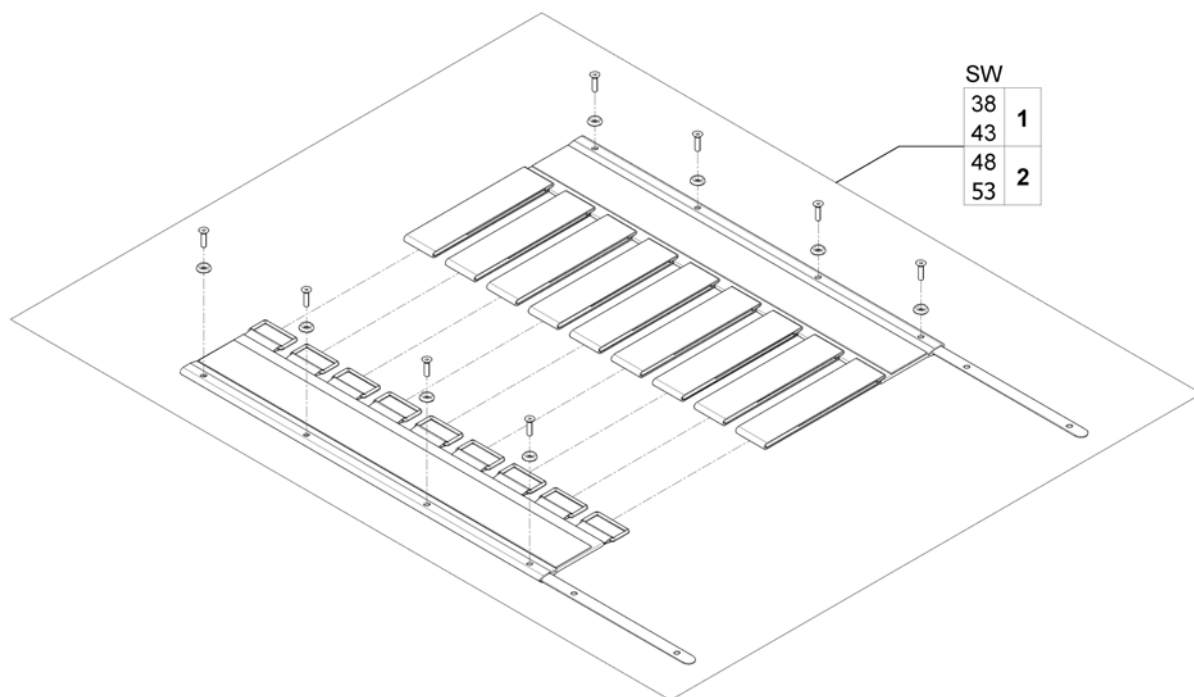


SW	
38	1
43	1
48	2
53	2

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557877	Sling Seat SW380/430, 510mm		1
2	SP1555911	Sling Seat SW480/530, 510mm		1

Telescopic Seat Frame and Universal Adapter


1581864 iso

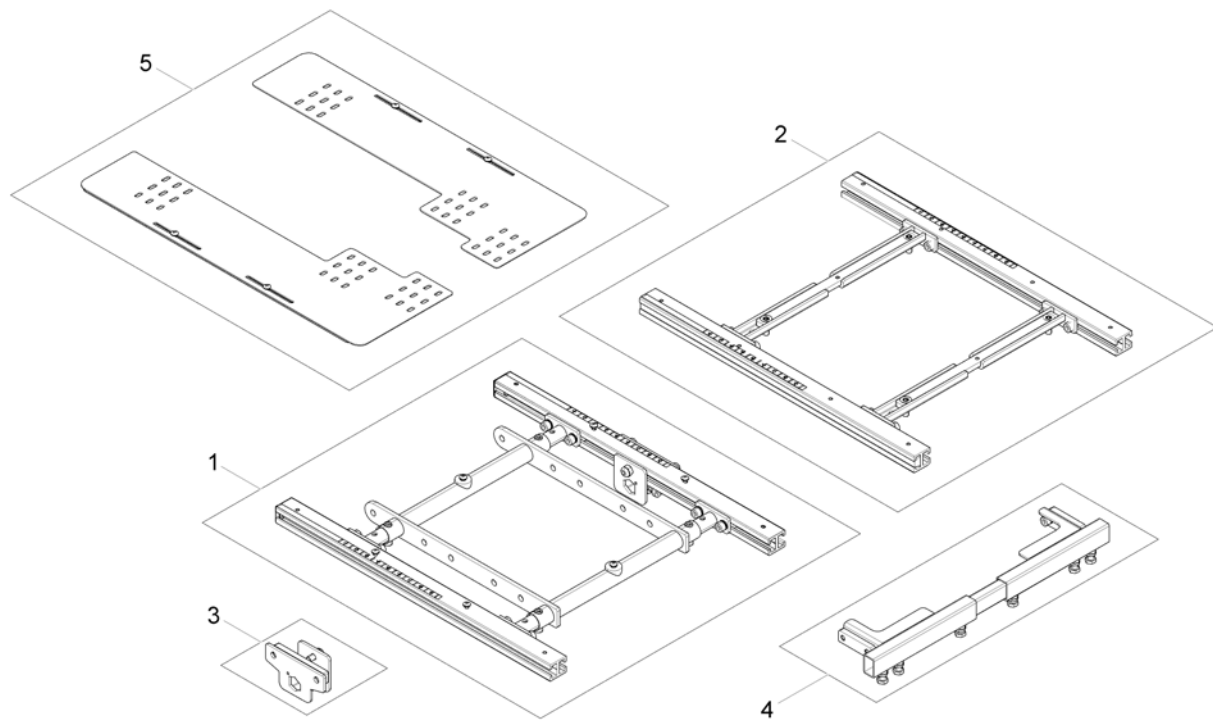


Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 12° / 20°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Adapter Kit for Universal Seat		1

Adapter Kit for Universal Seat

1581863.iso

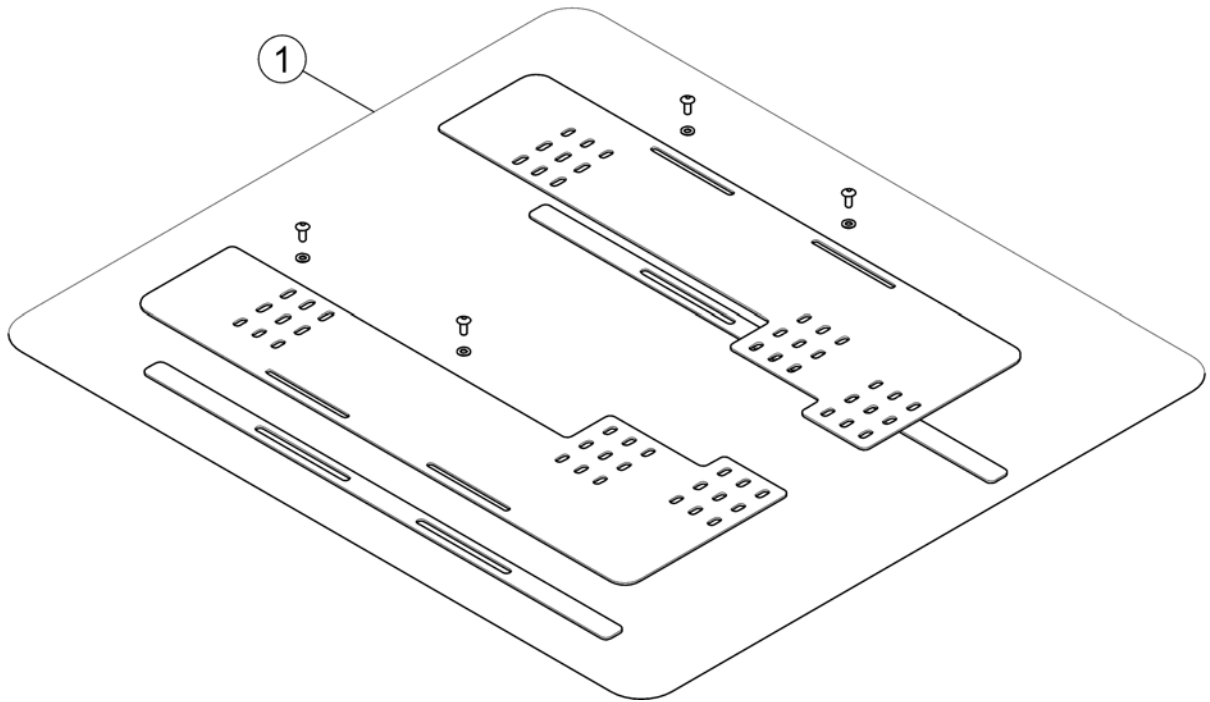

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580750	Adapter Kit for Universal Seat 510mm		1

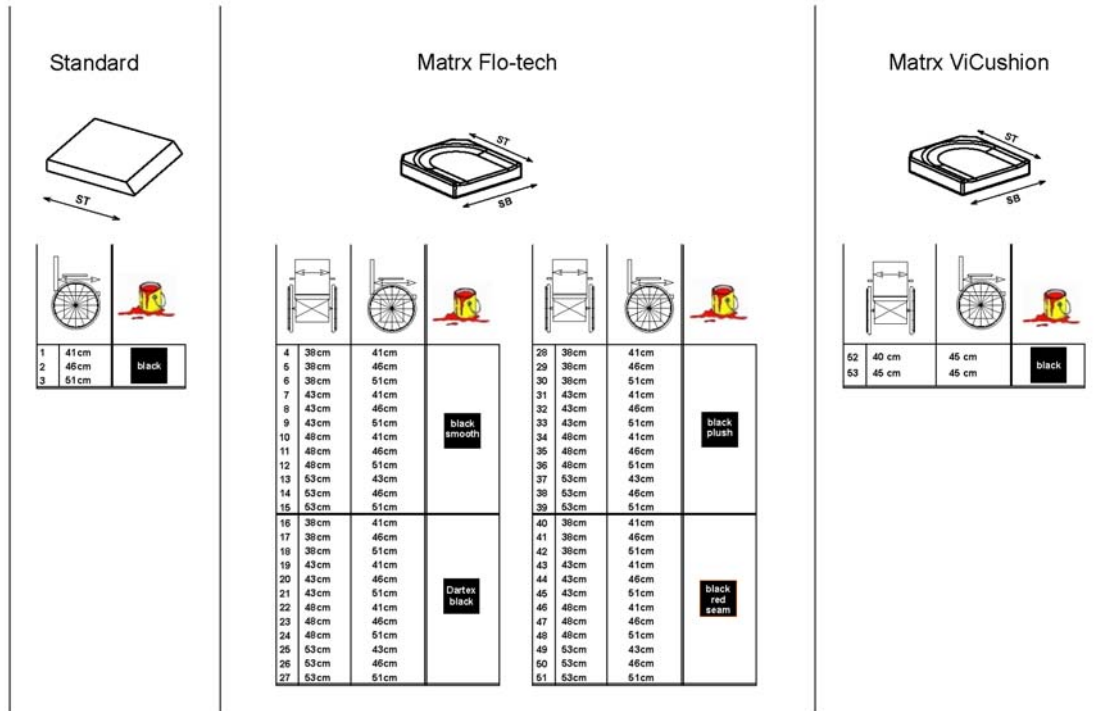

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard SD 410mm <i>[2141]</i>		1
2	1420461	Seat Cushion Standard SD 460mm <i>[2141]</i>		1
3	1420462	Seat Cushion Standard SD 510mm <i>[2141]</i>		1
4	SPFLB15/16	Flo-Tech Lite 15x16" <i>[2142+2151]</i>		1
5	SPFLB15/18	Seatcushion Easy-Adapt SB38 x ST46cm black smooth <i>[2142+2151]</i>		1
6	SPFLB15/20	Seatcushion Easy-Adapt SB38 x ST51cm black smooth <i>[2142+2151]</i>		1
7	SPFLB17/16	Seatcushion Easy-Adapt SB43 x ST41cm black smooth <i>[2142+2151]</i>		1
8	SPFLB17/18	Seatcushion Easy-Adapt SB43 x ST46cm black smooth <i>[2142+2151]</i>		1
9	SPFLB17/20	Seatcushion Easy-Adapt SB43 x ST51cm black smooth <i>[2142+2151]</i>		1
10	SPFLB19/16	Seatcushion Easy-Adapt SB48 x ST41cm black smooth <i>[2142+2151]</i>		1
11	SPFLB19/18	Seatcushion Easy-Adapt SB48 x ST46cm black smooth <i>[2142+2151]</i>		1
12	SPFLB19/20	Flo-Tech Lite 19x20" <i>[2142+2151]</i>		1
13	SPFCB21/17	Flo-Tech Contour 21x17" <i>[2142+2151]</i>		1
14	SPFCB21/18	Flo-Tech Contour 21x18" <i>[2142+2151]</i>		1
15	SPFCB21/20	Flo-Tech Contour 21x20" <i>[2142+2151]</i>		1
16	SP1529546-TR26	seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black <i>[2142+2152]</i>		1
17	SP1529547-TR26	seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black <i>[2142+2152]</i>		1
18	SP1529548-TR26	seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black <i>[2142+2152]</i>		1

Seat Cushion

Pos.	Part number	Description	Valid from → until	Qty
19	SP1529549-TR26	seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black <i>[2142+2152]</i>		1
20	SP1529550-TR26	seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black <i>[2142+2152]</i>		1
21	SP1529551-TR26	seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black <i>[2142+2152]</i>		1
22	SP1529552-TR26	seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black <i>[2142+2152]</i>		1
23	SP1529553-TR26	seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black <i>[2142+2152]</i>		1
24	SP1529554-TR26	seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black <i>[2142+2152]</i>		1
25	SP1529555-TR26	seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black <i>[2142+2152]</i>		1
26	SP1529556-TR26	seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black <i>[2142+2152]</i>		1
27	SP1529570-TR26	seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black <i>[2142+2152]</i>		1
28	SP1529546-TR35	seat cushion Flo-tech Lite SW 38 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
29	SP1529547-TR35	seat cushion Flo-tech Lite SW 38 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
30	SP1529548-TR35	seat cushion Flo-tech Lite SW 38 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
31	SP1529549-TR35	seat cushion Flo-tech Lite SW 43 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
32	SP1529550-TR35	seat cushion Flo-tech Lite SW 43 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
33	SP1529551-TR35	seat cushion Flo-tech Lite SW 43 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
34	SP1529552-TR35	Seat cushion Flo-tech Lite SW 48 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
35	SP1529553-TR35	seat cushion Flo-tech Lite SW 48 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
36	SP1529554-TR35	seat cushion Flo-tech Lite SW 48 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
37	SP1529555-TR35	seat cushion Flo-tech Contour SW 53 X SD 43 black chenille fabric <i>[2142+2154]</i>		1
38	SP1529556-TR35	seat cushion Flo-tech Contour SW 53 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
39	SP1529570-TR35	seat cushion Flo-tech Contour SW 53 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
40	SP1529546	Seatcushion Easy-Adapt SB38 x ST41cm black with red seam <i>[2142+2153]</i>		1
41	SP1529547	Seatcushion Easy-Adapt SB38 x ST46cm black with red seam <i>[2142+2153]</i>		1
42	SP1529548	Seatcushion Easy-Adapt SB38 x ST51cm black with red seam <i>[2142+2153]</i>		1
43	SP1529549	Seatcushion Easy-Adapt SB43 x ST41cm black with red seam <i>[2142+2153]</i>		1
44	SP1529550	Seatcushion Easy-Adapt SB43 x ST46cm black with red seam <i>[2142+2153]</i>		1
45	SP1529551	Seatcushion Easy-Adapt SB43 x ST51cm black with red seam <i>[2142+2153]</i>		1
46	SP1529552	Seatcushion Easy-Adapt SB48 x ST41cm black with red seam <i>[2142+2153]</i>		1
47	SP1529553	Seatcushion Easy-Adapt SB48 x ST46cm black with red seam <i>[2142+2153]</i>		1
48	SP1529554	Seatcushion Easy-Adapt SB48 x ST51cm black with red seam <i>[2142+2153]</i>		1
49	SP1529555	Seatcushion Easy-Adapt SB53 x ST43cm black with red seam <i>[2142+2153]</i>		1
50	SP1529556	Seatcushion Easy-Adapt SB53 x ST46cm black with red seam <i>[2142+2153]</i>		1

Pos.	Part number	Description	Valid from → until	Qty
51	SP1529570	Seatacushion Easy-Adapt SB53 x ST51cm black with red seam <i>[2142+2153]</i>		1
52	SP1531760	Seat Cushion MatrX Vi SW 40 x SD 45cm (16" x 18") <i>[2144]</i>		1
53	SP1531761	Seat Cushion MatrX Vi SW 45 x SD 45cm (18" x 18") <i>[2144]</i>		1
54	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm <i>[2143]</i>		1

Backrest

SPDKTE20 160




Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest for Standard Seat		1
2	-	Backrest for Flex2 Seat		1
3	-	Backrest for Max Seat		1
4	-	Backrest for Modulite Seat		1

Backrest for Standard Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Backrest Standard Seat, quick release [0958]		1
-		Backrest Standard Seat, settable [0959]		1
-		Backrest Standard Seat, Powered Recliner [0947]		1

Backrest Standard Seat, quick release


1523521PAC.dwg

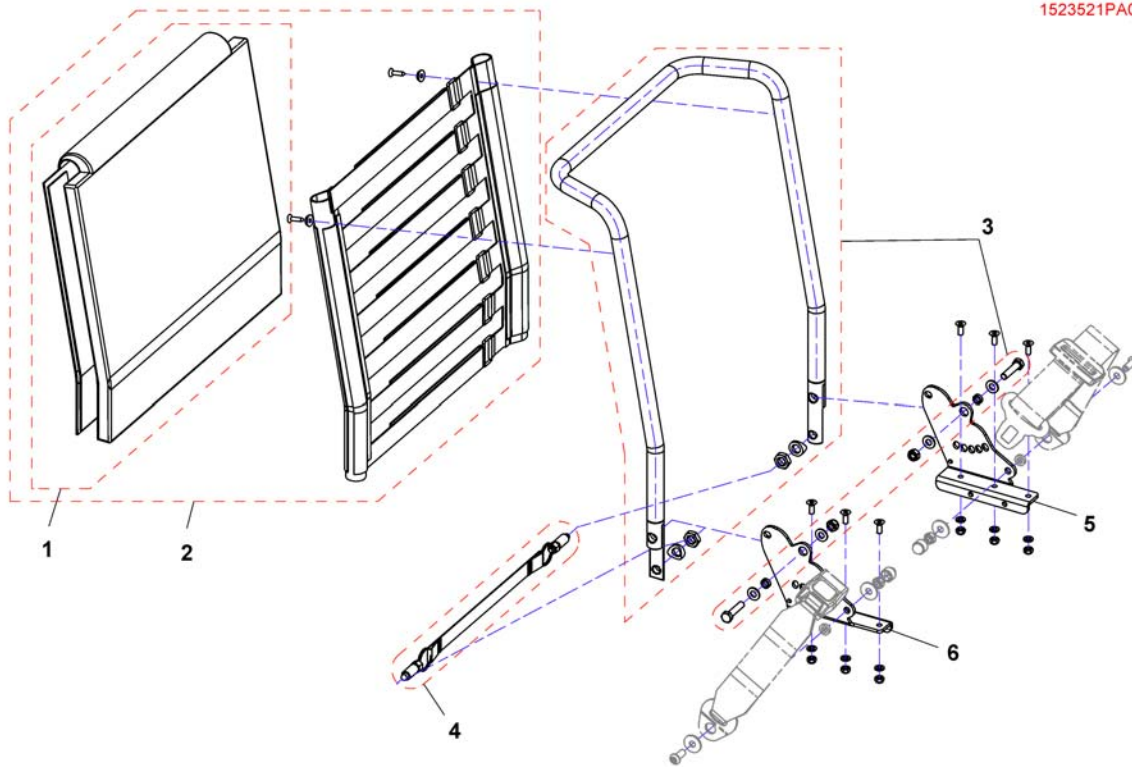

Pos.	Part number	Description	Valid from → until	Qty
1	1420445	Backrest upholstery, 480mm		1
1	1420452	Backrest upholstery, 540mm		1
2	1422062	Sling Back with Backrest upholstery 480 mm		1
2	1422063	Sling Back with Backrest upholstery 540mm		1
3	SP1523520	Backrest Tube 480mm		1
3	SP1527953	Backrest Tube 540mm		1
4	SP1528346	Belt with snappers and mounting hardware		1
5	SP1523519	Backrest bracket LH		1
6	SP1523518	Backrest bracket RH		1

Backrest Standard Seat, settable


1422061X.iso

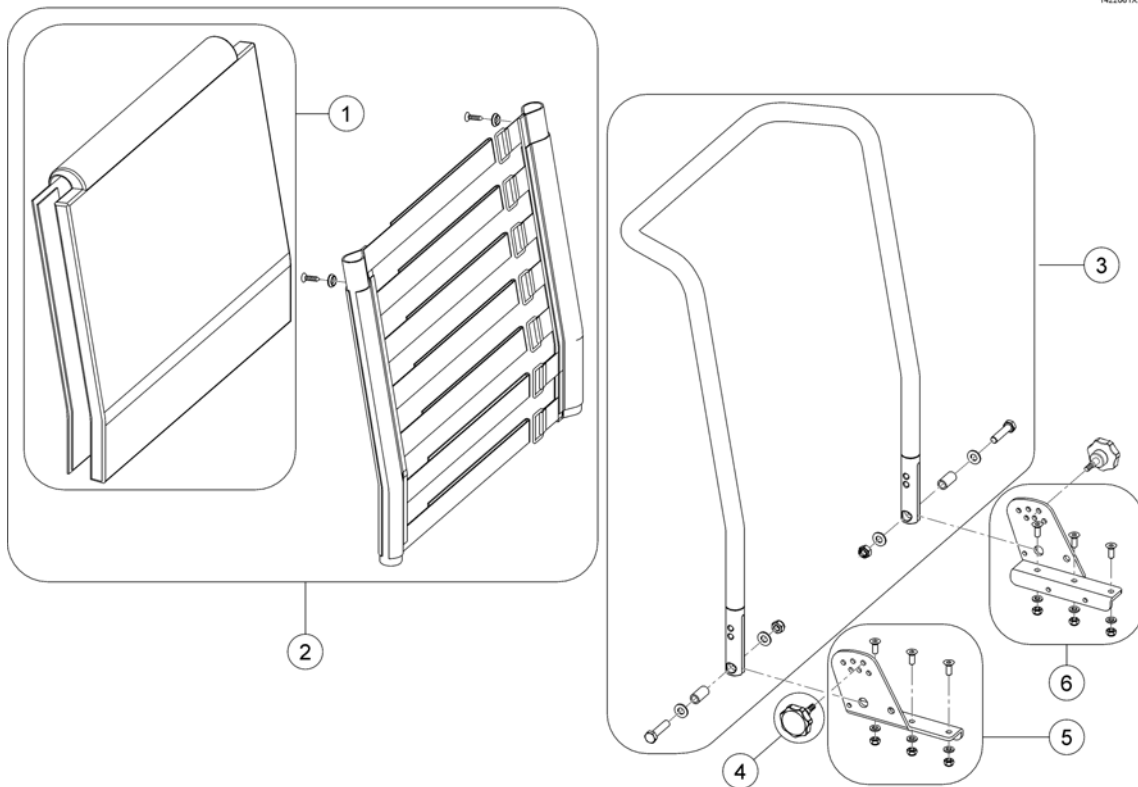


Pos.	Part number	Description	Valid from → until	Qty
1	1420445	Backrest upholstery, 480mm		1
1	1420452	Backrest upholstery, 540mm		1
2	1422062	Sling Back with Backrest upholstery 480 mm		1
2	1422063	Sling Back with Backrest upholstery 540mm		1
3	SP1419996	Backrest Frame 48cm height		1
3	SP1420465	Backrest Frame 54cm height		1
4	SP1420444	Star Grip M8		1
5	SP1541567	Backrest bracket RH		1
6	SP1541568	Backrest bracket LH		1

Backrest Standard Seat, Powered Recliner

1531008K.dwg

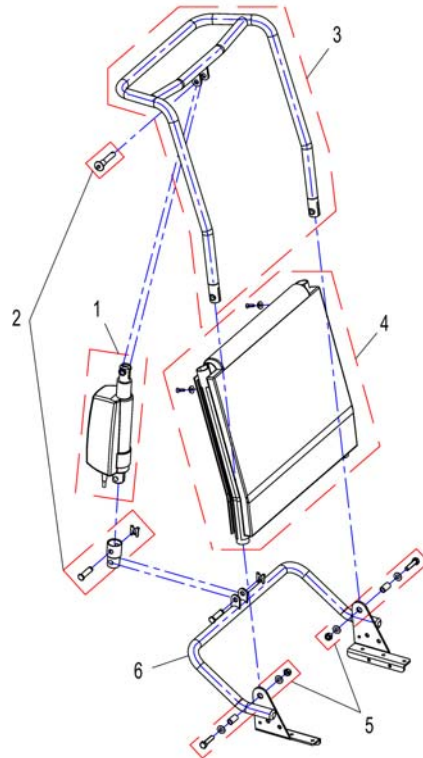


Pos.	Part number	Description	Valid from → until	Qty
1	SP1533541	Actuator Backrest adjustment ACS2		1
1	SP1533542	Actuator Backrest adjustment VR2 & R-NET		1
1	SP1446520	Actuator ACS2 / Shark <i>for Shark!</i>		1
2	1446700	Mounting Kit Actuator		1
3	1442864-9005	Backrest frame for 480 mm height (push handle), adjustable		1
3	1442866-9005	Backrest frame for 540 mm height (push handle), adjustable		1
4	1422062	Sling Back with Backrest upholstery 480 mm		1
4	1422063	Sling Back with Backrest upholstery 540mm		1
5	1446699	Fixation Kit for backrest tube		1
6	1442845-9005	Lower Bracket for backrest frame		1

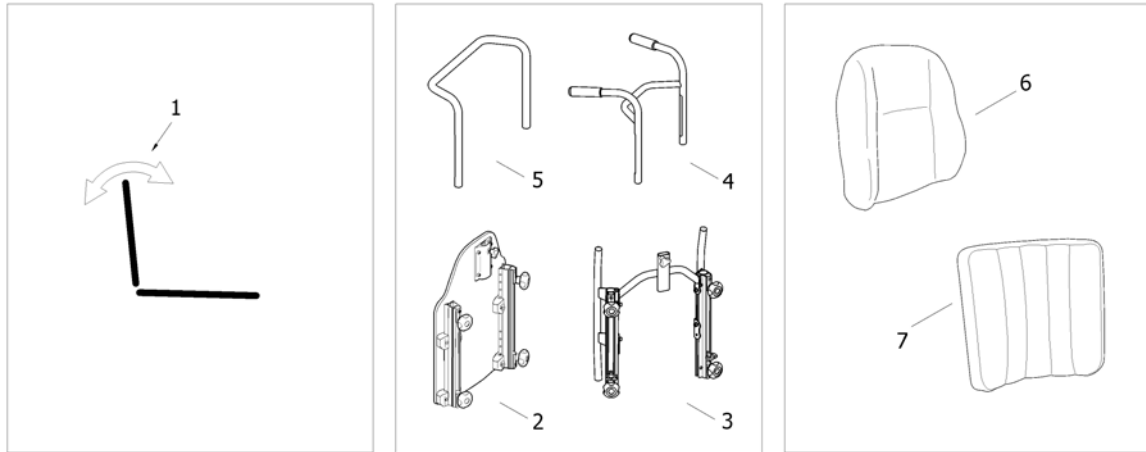

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest Recliner		1
2	-	Backrest Flex2		1
3	-	Backrest frame Flex2, tension adjustable		1
4	-	Push handles Flex2 [1306]		1
5	-	Push handles Flex2, attendant control [1306]		1
6	-	Backrest cushion		1
7	-	Tension adjustable		1


Backrest Recliner

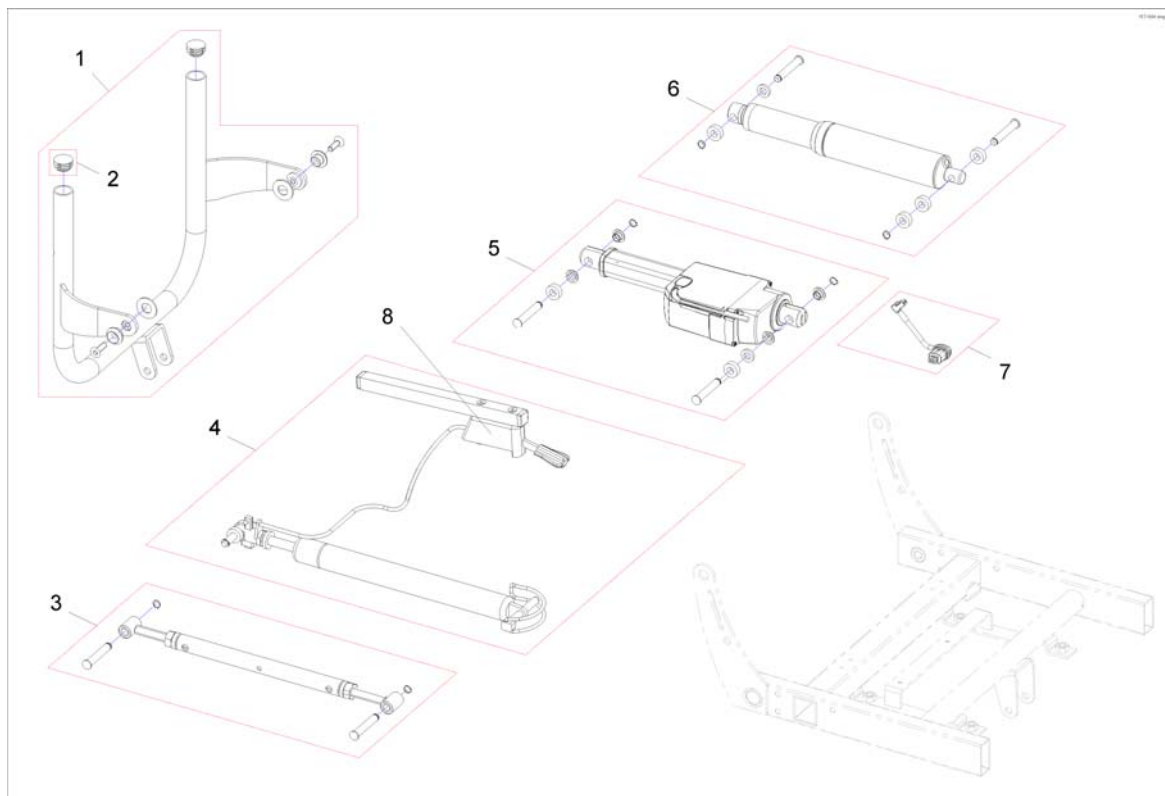

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531265	Backrest Support Flex2, SW 39 cm		1
1	SP1531266	Backrest Support Flex2, SW 44 cm		1
1	SP1531267	Backrest Support Flex2, SW 49 cm		1
2	L01008	Tube End Plug SI 22		1
3	SP1531270	Backrest adjustment, manual [0959]		1
4	SP1552566	Backrest adjustment, pneumatic		1
5	SP1552547	Backrest adjustment, LA23 powered, 30° [0947]	2011-11-09 →	1
6	SP1531268	Backrest adjustment, powered ACS2 [0947]	→ 2011-11-08	1
7	SP1552297	Actuator Cable LA23 Shark (~500mm) [0947]	2011-11-09 →	1
8	N368003	Release Lever		1
	SP1524744	Adaptercable Actuator AMP -> DX2 [0947+1021]		1

Backrest Flex2


152694PAC.dwg

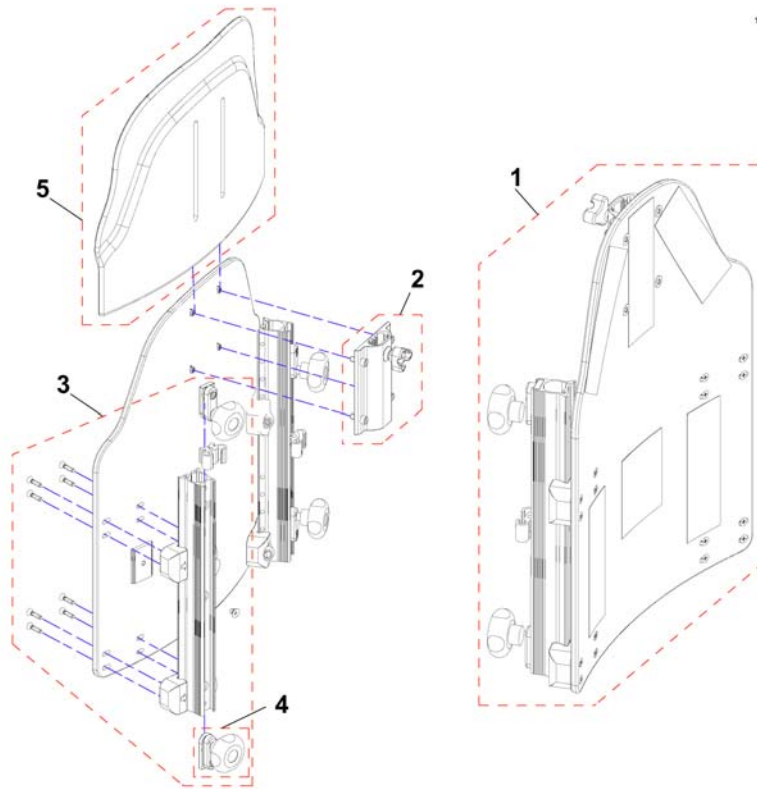

Pos.	Part number	Description	Valid from → until	Qty
1	1440151	BACKREST PLATE Flex2 405X485 CPL		1
1	1440152	BACKREST PLATE Flex2 455X485 CPL		1
1	1440153	BACKREST PLATE Flex2 505X485 CPL		1
2	1446010	Bracket for Neck-/Headrest Flex2		1
3	1446008	Rail RH, backrest FLEX2, fixed		1
3	1446009	Rail LH, backrest FLEX2, fixed		1
4	5253088	Fixation block		1
5	1440147	EXTENDER 405X290 CPL, BACKREST FLEX2		1
5	1440148	EXTENDER 455X290 CPL, BACKREST FLEX2		1
5	1440149	EXTENDER 505X290 CPL, BACKREST FLEX2		1


Backrest frame Flex2, tension adjustable

1445697PAC.iso

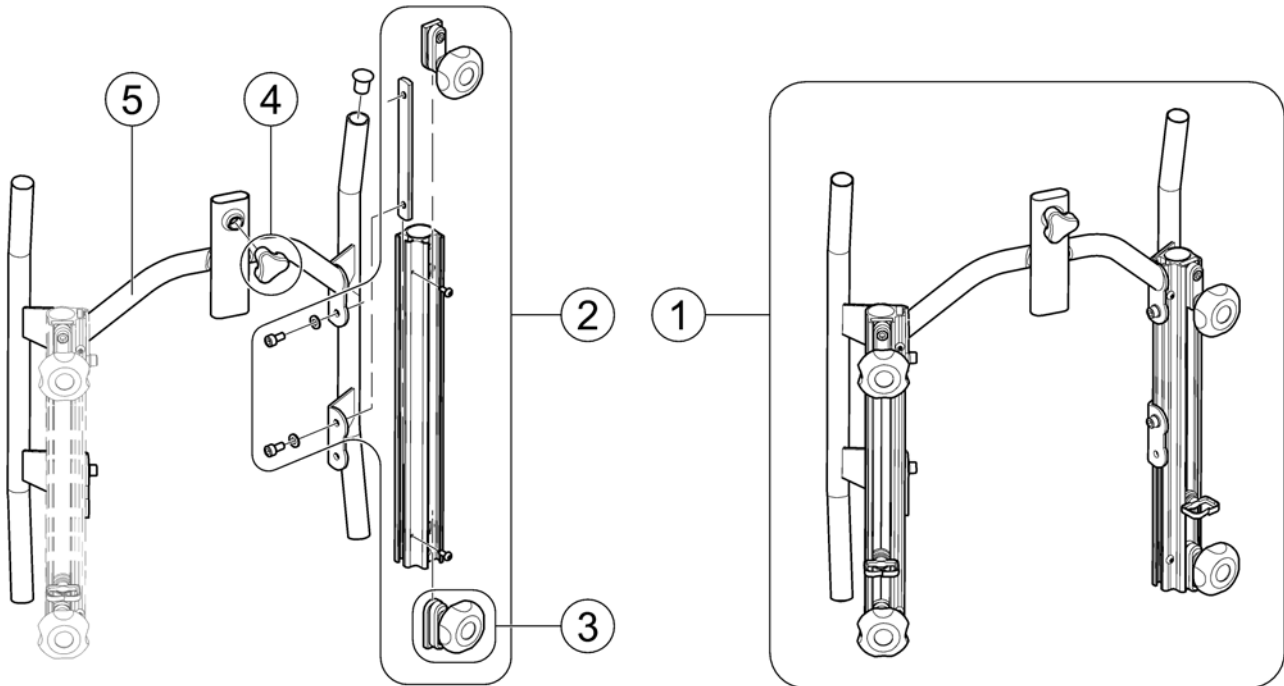

Pos.	Part number	Description	Valid from → until	Qty
1	1445697	Backrest, adjustable 292 (SW 390mm), cpl.		1
1	1445698	Backrest, adjustable 342 (SW 440mm), cpl.		1
1	1445699	Backrest, adjustable 392 (SW 490mm), cpl.		1
2	1445848	Backrest Rail, cpl.		1
3	5253088	Fixation block		1
4	1444175	Handwheel M12		1
5	1445844-0509	Backrest Frame SW 390mm		1
5	1445845-0509	Backrest Frame SW 440mm		1
5	1445846-0509	Backrest Frame SW 490mm		1


1526957PAC.dwg

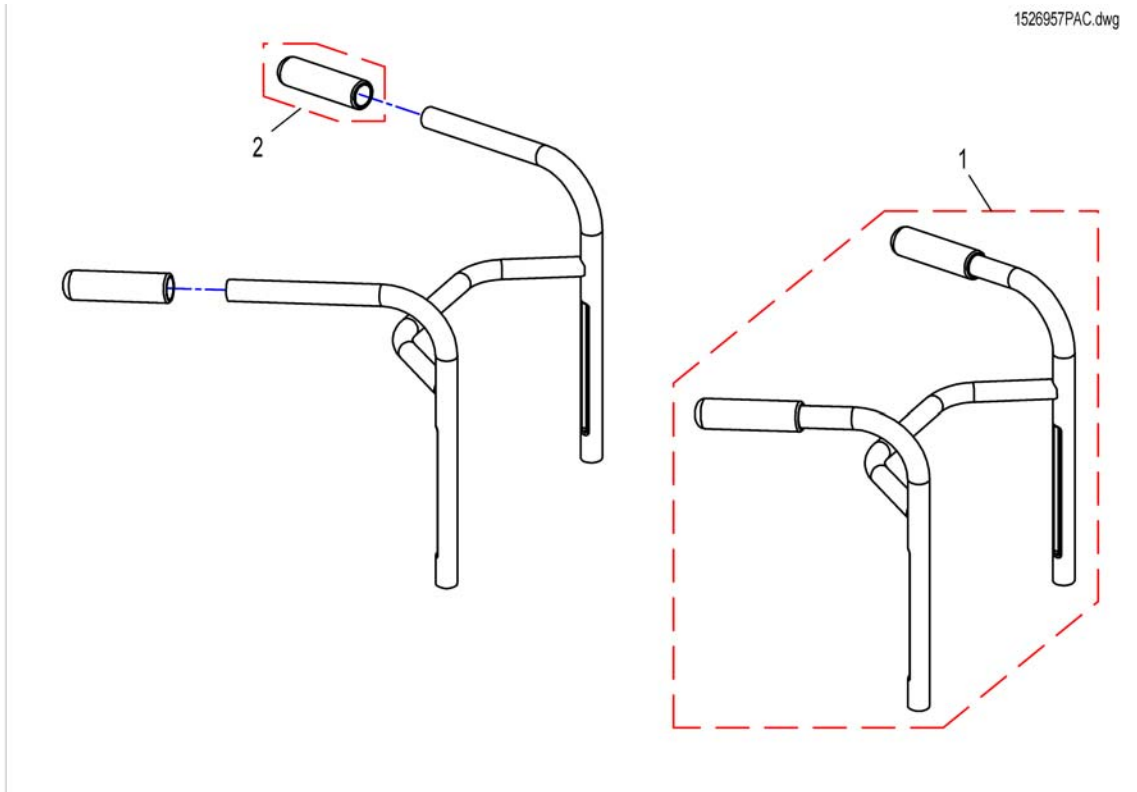

Pos.	Part number	Description	Valid from → until	Qty
1	SP1526957	Push Handle SW39		1
1	SP1526958	Push Handle SW44		1
1	SP1526959	Push Handle SW49		1
2	1435421	Handle plastic black Ø22 "Invacare"		1

Push handles Flex2, attendant control



1534551X.dwg

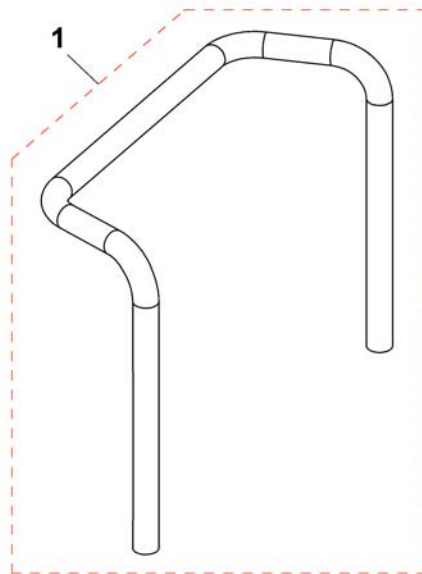

Pos.	Part number	Description	Valid from → until	Qty
1	1425840-9005	PUSH HANDLE SB 39		1
1	1425839-9005	PUSH HANDLE SW 44		1
1	SP1534553	PUSH HANDLE SB 49		1


1526905PAC.dwg

FLEX II

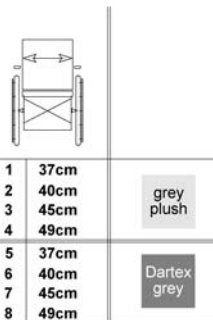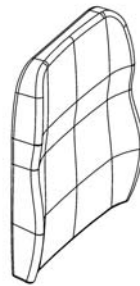

Pos.	Part number	Description	Valid from → until	Qty
6	-	No longer available - please refer to Replacement Info for which part is to be used instead. <i>SP1427119-TR26 Dartex black or 1435798 Grey Plush</i>		1
7	-	No longer available - please refer to Replacement Info for which part is to be used instead. <i>SP1427120-TR26 Dartex black or 1444340 Grey Plush</i>		0
1	1444339	Backrest Cushion Laguna, 372x550mm, Grey Plush		1
2	1435798	Backrest Cushion Laguna, 402x550mm, Grey Plush		1
3	1444340	Backrest Cushion Laguna, 452x550mm, Grey Plush		1
4	1444341	Backrest Cushion Laguna, 492x550mm, Grey Plush		1
5	1427118	Backrest cushion comfort Laguna (XS), 372x550 mm, Dartex grey (TR23)		1
8	1427121	Backrest Cushion Laguna, 492x550mm, Dartex Grey		1

Tension adjustable

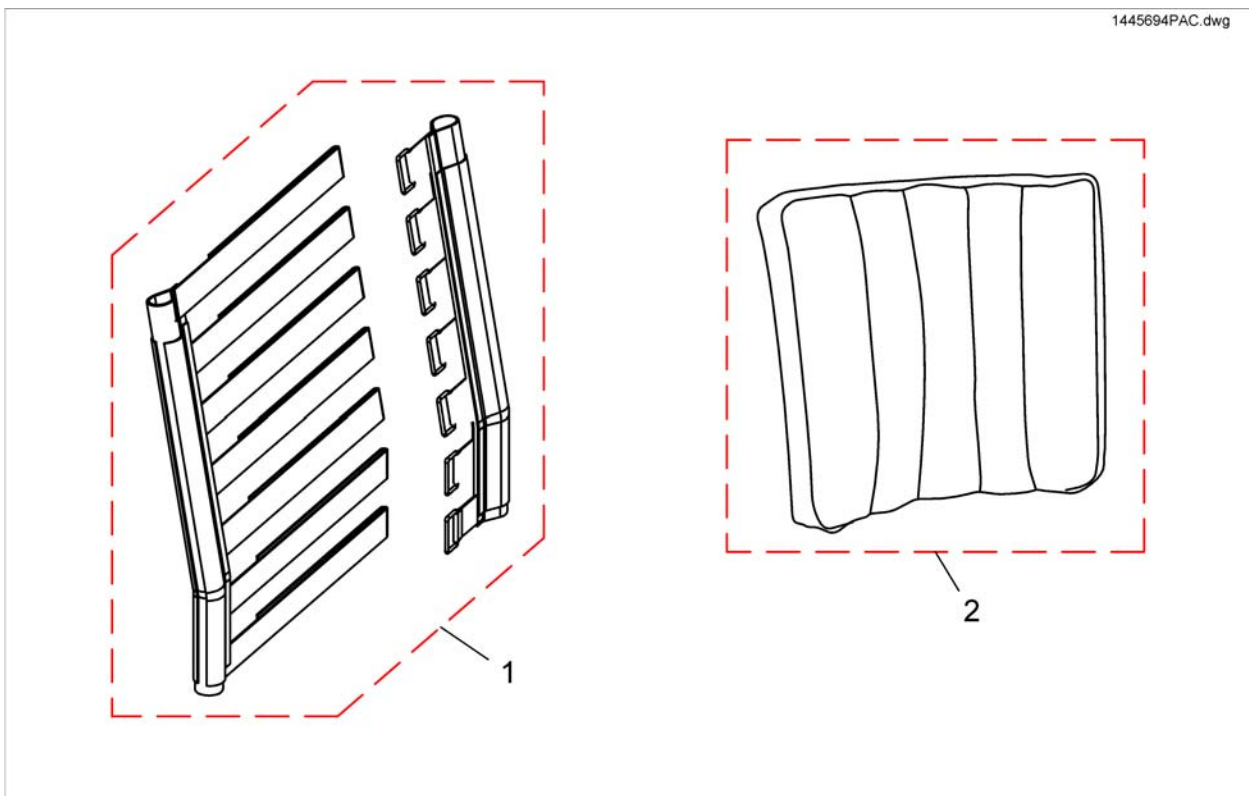


Pos.	Part number	Description	Valid from → until	Qty
1	1445694	Backrest, tension adjustable, SW up to 49cm		1
2	1444344	Cover SW39 TR32		1
2	1444345	Cover SW44 TR32		1
2	1444346	Cover SW49 TR32		1
2	1446512	Cover SW39, black plush		1
2	1446513	Cover SW44, black plush		1
2	1446522	Cover SW49, black plush		1
2	1508865	Cover SW39, dartex grey		1
2	1508866	Cover SW44, dartex grey		1
2	1508867	Cover SW49, dartex grey		1

Backrest for Max Seat


Pos.	Part number	Description	Valid from → until	Qty
-		Backrest Recliner		1
-		Backrest Max-Seat		1

Backrest Recliner

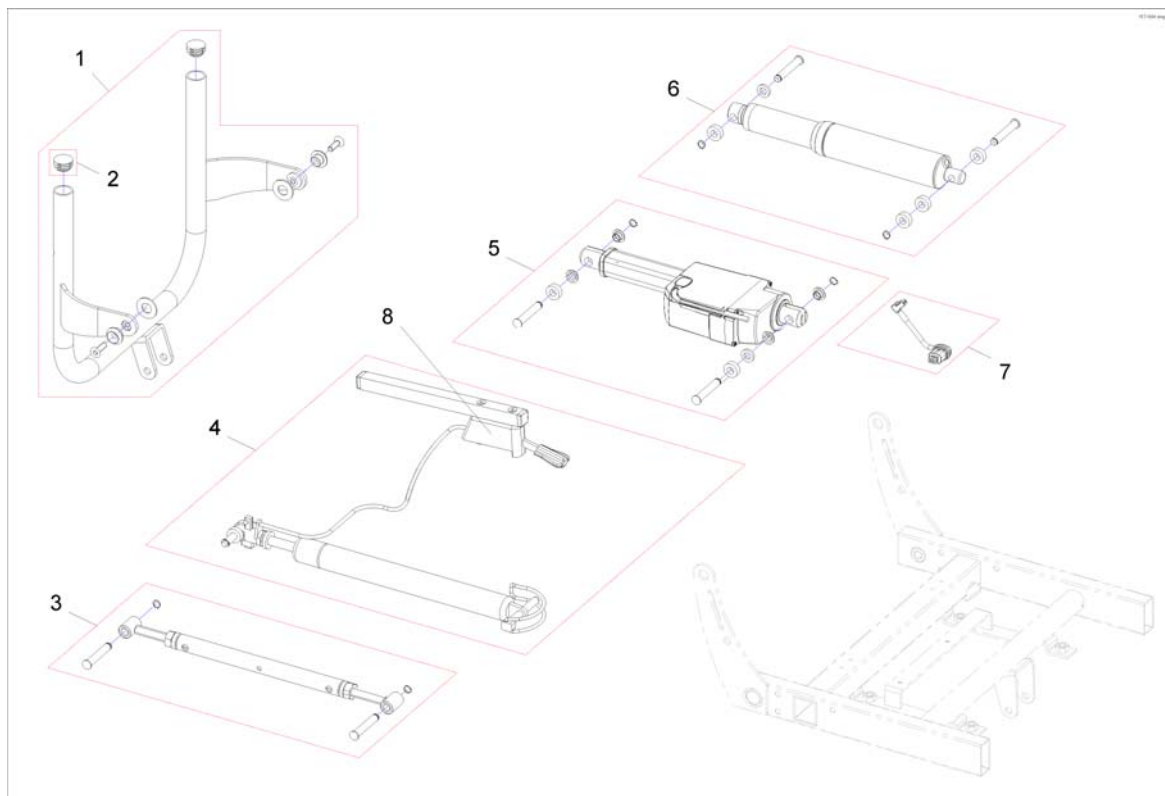

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551684	Backrest Support Flex2, SW 55 cm (with and without lifter)		1
2	L01008	Tube End Plug SI 22		1
3	SP1531270	Backrest adjustment, manual [0959]		1
4	SP1552566	Backrest adjustment, pneumatic		1
5	SP1552547	Backrest adjustment, LA23 powered, 30° [0947]	2011-11-09 →	1
6	SP1531268	Backrest adjustment, powered ACS2 [0947]	→ 2011-11-08	1
7	SP1552297	Actuator Cable LA23 Shark (~500mm) [0947]	2011-11-09 →	1
8	N368003	Release Lever		1
	SP1524744	Adaptercable Actuator AMP -> DX2 [0947+1021]		1


1526824X.dwg

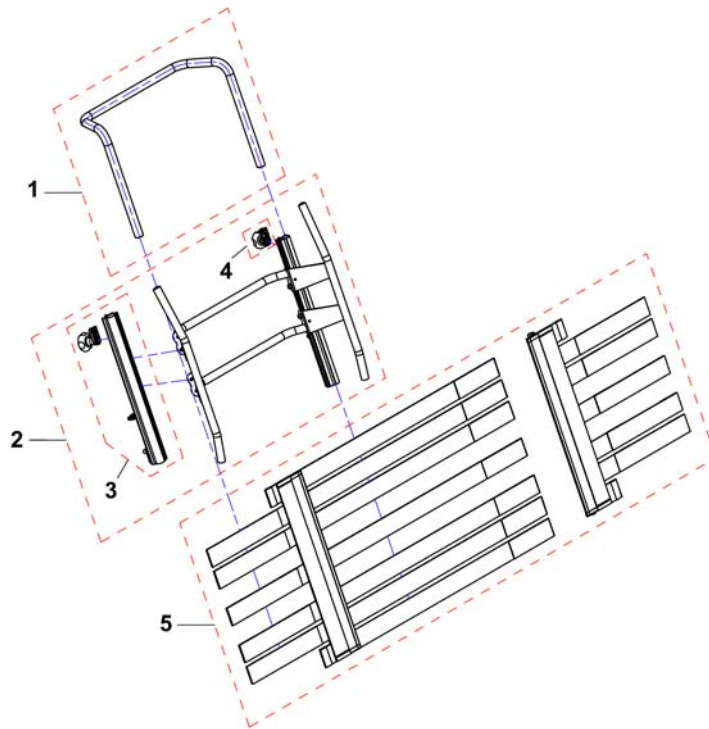

Pos.	Part number	Description	Valid from → until	Qty
1	SP1543081	Support tube, SW55		1
2	SP1526824	Backrest frame, cpl., SW55		1
3	1445848	Backrest Rail, cpl.		1
4	5253088	Fixation block		1
5	SP1526308	Backrest, tension adjustable, SW 55 - 61cm		1
-	1526459	Backrest upholstery SW 55 cm		1

Backrest for Modulite Seat


1586095.000

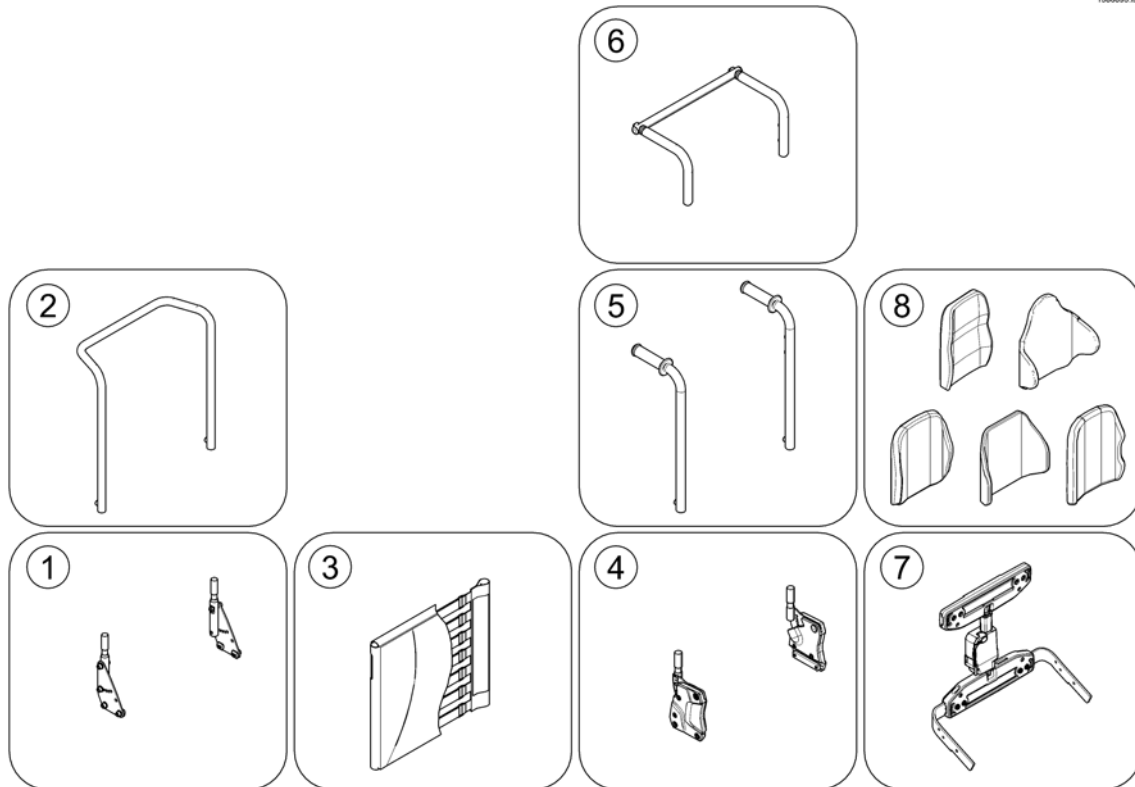

Pos.	Part number	Description	Valid from → until	Qty
1	-	Support for Backrest Frame <i>[0975+2161]</i>		0
2	-	Backrest Frame <i>[2161]</i>		0
3	-	Sling Back, tension adjustable <i>[2171]</i>		0
4	-	Backrest support		0
5	-	Backposts <i>[2162/2163]</i>		0
6	-	Connection tube <i>[1307]</i>		0
7	-	Back Recliner / Stabilization bracket <i>[0977]</i>		0
8	-	Backrest Cushions <i>[2172/2173/2174/2175/2176/2177/2178]</i>		1

Support for Backrest Frame


1571553 iso

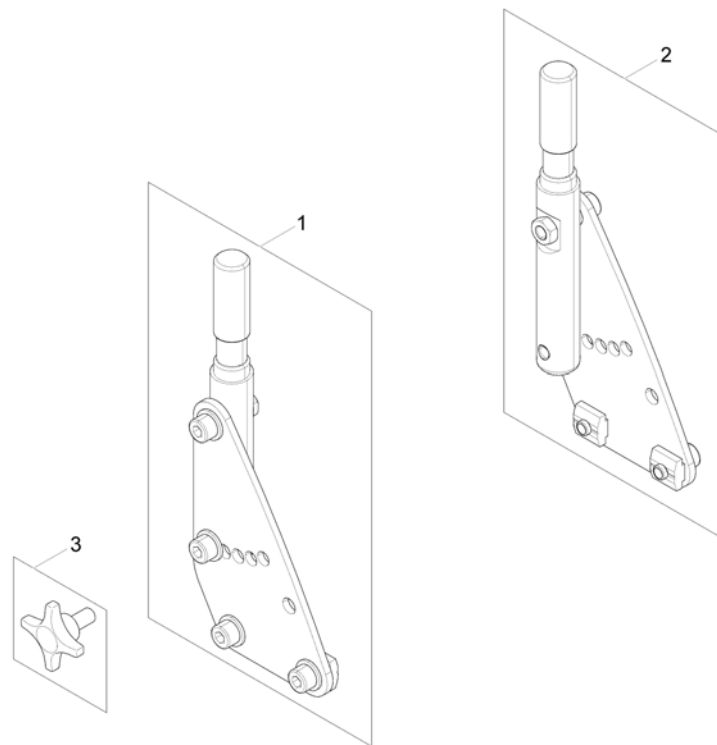

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571542	Backrest Support RH		1
2	SP1571553	Backrest Support LH		1
3	5824117	hand wheel M8x15 <i>(only swedish homologation)</i>		2

Backrest Frame


1571831.00

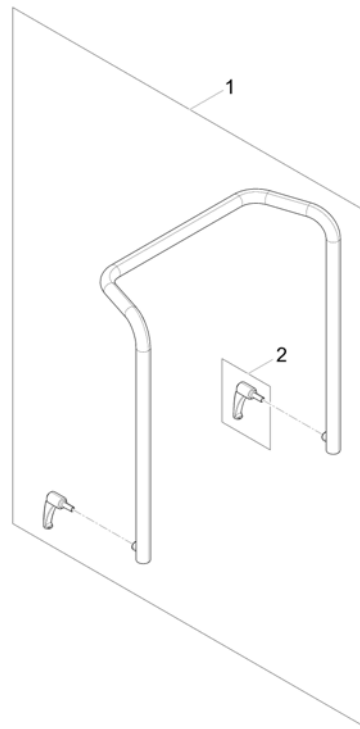

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567810	Backrest Frame		1
2	0021-70002	Indexable lever 63mm M8x16		1

Sling Back, tension adjustable


1571829.indd

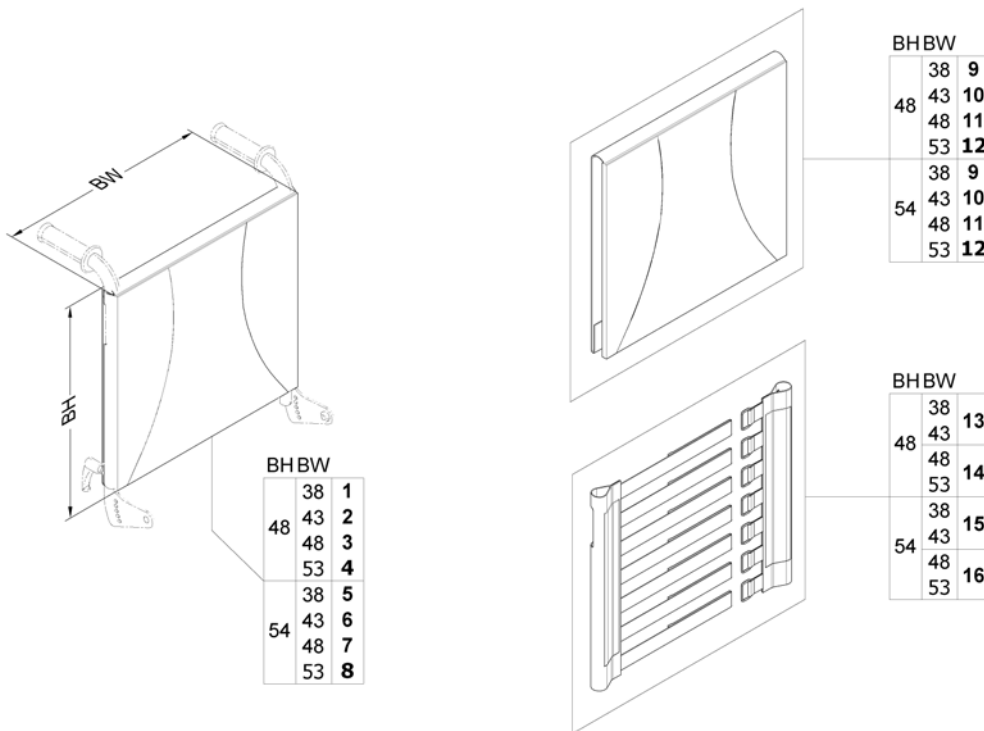

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567529	Sling Back, tension adjustable & Upholstery, BW 380mm / BH 480mm, cpl.		1
2	SP1567465	Sling Back, tension adjustable & upholstery, BW 430mm / BH 480mm, cpl.		1
3	SP1567530	Sling Back, tension adjustable & upholstery, BW 480mm / BH 480mm, cpl.		1
4	SP1567531	Sling Back, tension adjustable & upholstery, BW 530mm / BH 480mm, cpl.		1
5	SP1567627	Sling Back, tension adjustable & upholstery, BW 380mm / BH 540mm, cpl.		1
6	SP1567746	Sling Back, tension adjustable & upholstery, BW 430mm / BH 540mm, cpl.		1
7	SP1567747	Sling Back, tension adjustable & upholstery, BW 480mm / BH 540mm, cpl.		1
8	SP1567748	Sling Back, tension adjustable & upholstery, BW 530mm / BH 540mm, cpl.		1
9	SP1563563	Backrest upholstery		1
10	SP1563653	Backrest upholstery		1
11	SP1563654	Backrest upholstery		1
12	SP1563655	Backrest upholstery		1
13	SP1571492	Sling back		1
14	SP1571493	Sling back		1
15	SP1571494	Sling back		1
16	SP1571510	Sling back		1


Backrest support

1572928 Rev

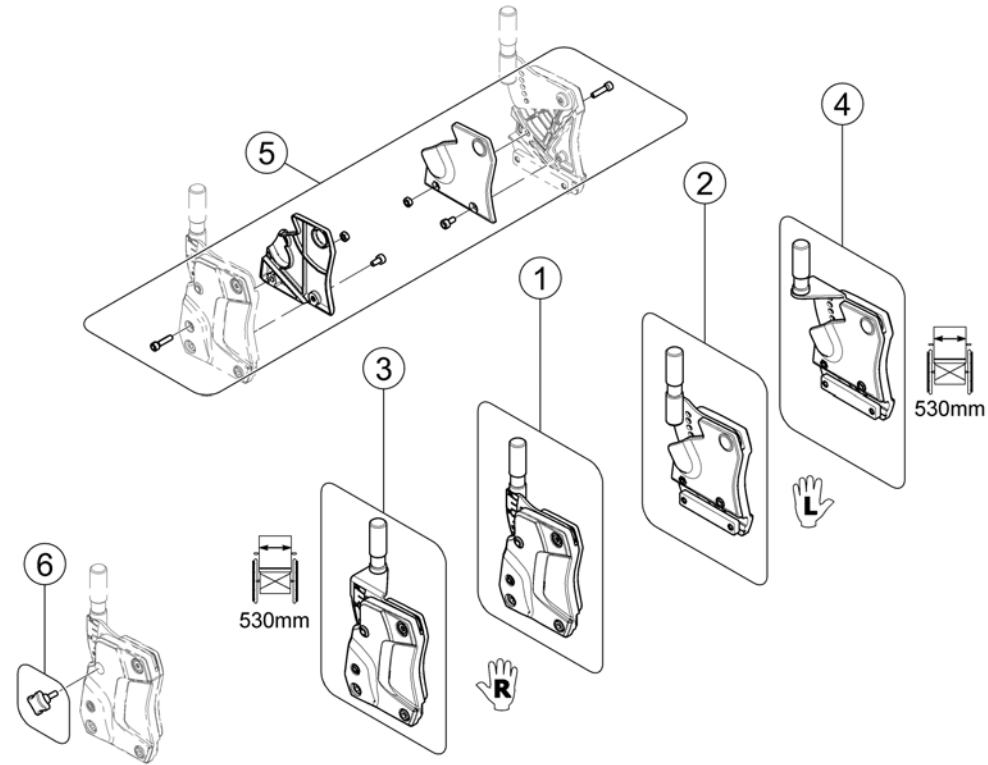

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582709	Backrest Support - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571560</i>		1
2	SP1582708	Backrest Support - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571561</i>		1
3	SP1582952	Backrest Support SW 530mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571568</i>		1
4	SP1582951	Backrest Support SW 530mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571569</i>		1
5	SP1571676	Cover		1
6	SP1579877	Hand Screw M6 <i>(Swedish Homologation only!)</i>	→ 2015-07-31	2
6	SP1589324	Cross handle M8, cpl. <i>(Swedish Homologation only!)</i>	2015-08-01 →	2


Backposts

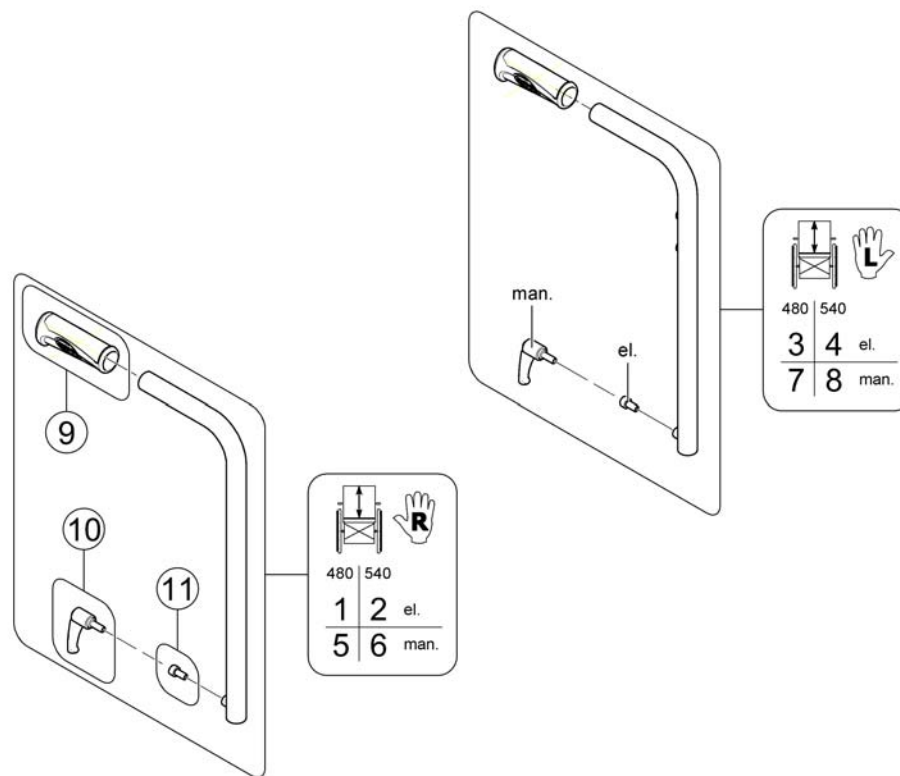

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587177	Backpost el. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563102</i>		1
2	SP1587184	Backpost el. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563103</i>		1
3	SP1587176	Backpost el. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1560576</i>		1
4	SP1587183	Backpost el. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1559578</i>		1
5	SP1587163	Backpost man. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572489</i>		1
6	SP1587182	Backpost man. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572490</i>		1
7	SP1587162	Backpost man. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572491</i>		1
8	SP1587178	Backpost man. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572492</i>		1
9	SP1586122	Handles (pair) <i>replaces item SP1563651</i>		1
10	0021-70002	Indexable lever 63mm M8x16		1
11	1567999	Bolt M8x16 8.8 black DIN912		1

Connection tube


1571835 no

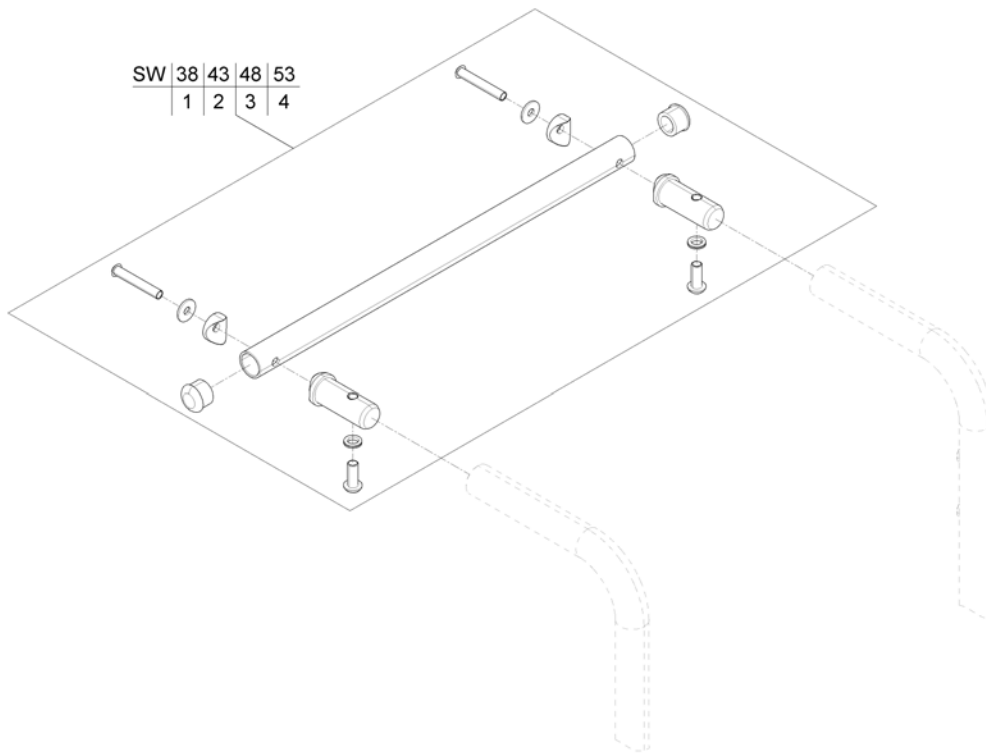

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570370	Connection tube		1
2	SP1570371	Connection tube		1
3	SP1570372	Connection tube		1
4	SP1570373	Connection tube		1

Back Recliner / Stabilization bracket


1571097.kim

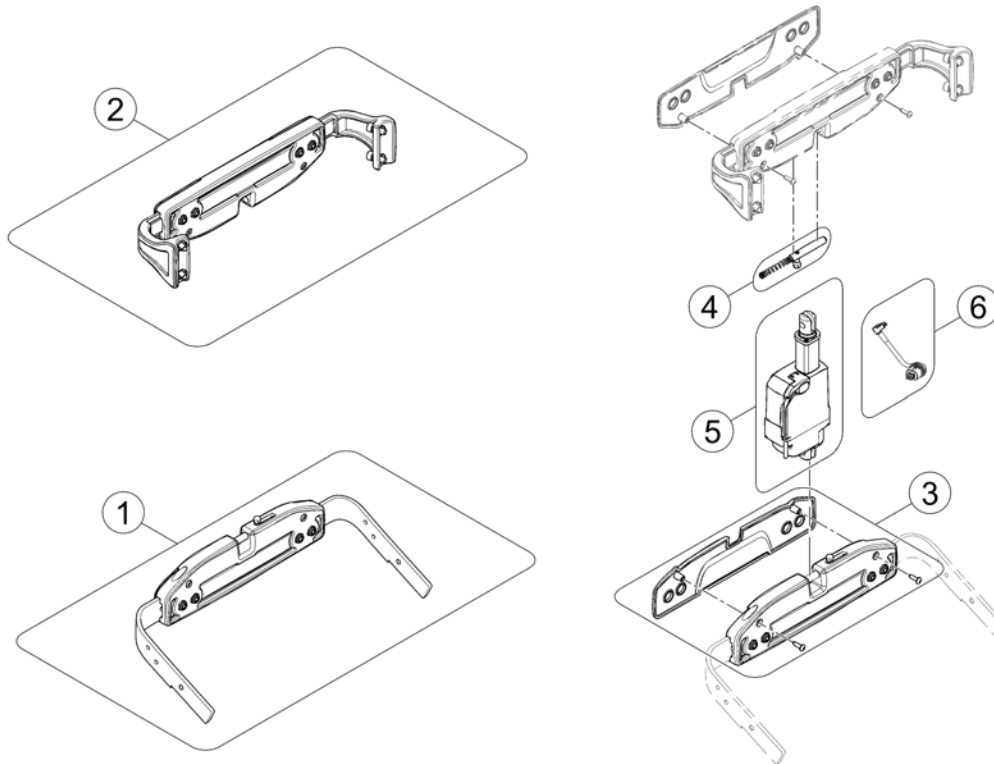

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567169	Actuator bracket (pre set to SW 43cm)		1
2	SP1567168	Stabilization bracket (pre set to SW 43cm)		1
3	SP1571702	Cover		1
4	SP1571704	Locking bolt		1
5	SP1557820	Actuator		1
6	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1

Backrest Cushions


1588915.iss

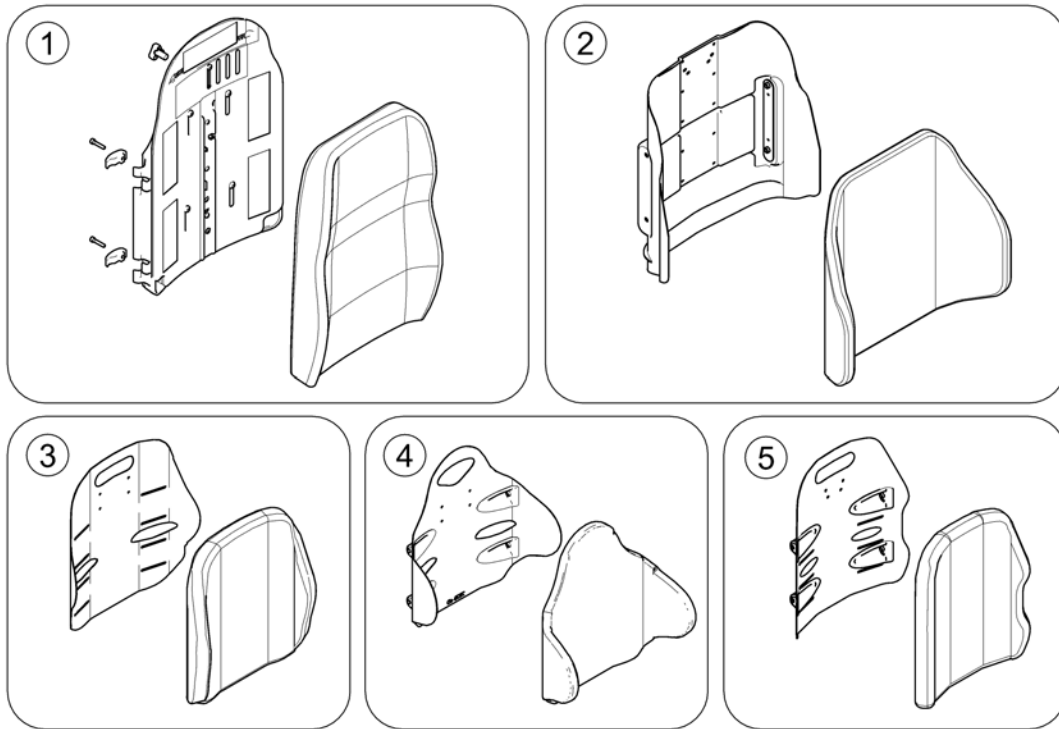

Pos.	Part number	Description	Valid from → until	Qty
1	-	Flex 3 / Laguna		1
2	-	Matrx Posture Back		1
3	-	Matrx Elite T/R Back	2015-07-01 →	1
4	-	Matrx Elite Deep Back	2015-07-01 →	1
5	-	Matrx Elite (Full) Back		1


Flex 3 / Laguna

1573106.mw

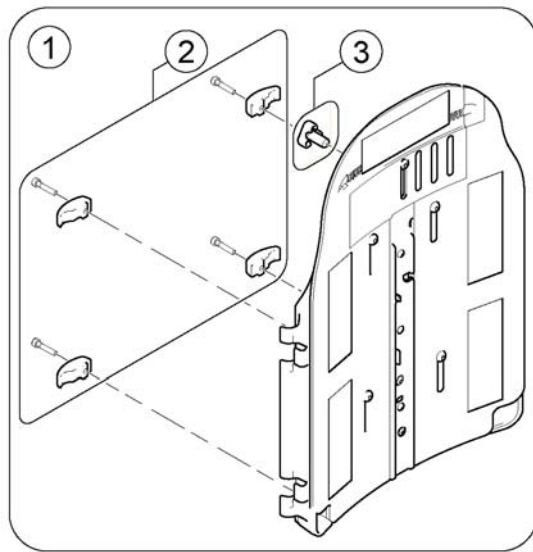

Flex 3

Laguna

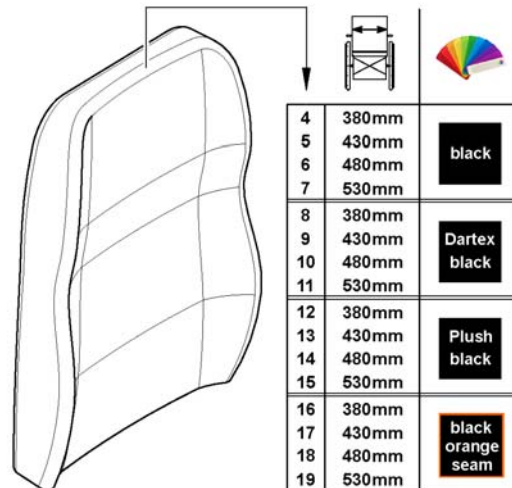

Pos.	Part number	Description	Valid from → until	Qty
1	SP1568489	Flex3 backrest plate <i>[2172]</i>		1
2	SP1572321	Clamps		1
3	1497411	HANDWHEEL M12 LONG NECK/HEADREST		1
4	SP1529585	Backrest Cushion Laguna / EasyAdapt, black - SW 380mm <i>[2173+2151]</i>		1
5	SP1529586	Backrest Cushion Laguna / EasyAdapt, black - SW 430mm <i>[2173+2151]</i>		1
6	SP1529587	Backrest Cushion Laguna / EasyAdapt, black - SW 480mm <i>[2173+2151]</i>		1
7	SP1529596	Backrest Cushion Laguna / EasyAdapt, black - SW 530mm <i>[2173+2151]</i>		1
8	SP1529538-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 380mm <i>[2173+2152]</i>		1
9	SP1529539-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 430mm <i>[2173+2152]</i>		1
10	SP1529540-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 480mm <i>[2173+2152]</i>		1
11	SP1529545-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 530mm <i>[2173+2152]</i>		1
12	SP1529538-TR35	Backrest Cushion Laguna, Plush black - SW 380mm <i>[2173+2154]</i>		1
13	SP1529539-TR35	Backrest Cushion Laguna, Plush black - SW 430mm <i>[2173+2154]</i>		1
14	SP1529540-TR35	Backrest Cushion Laguna, Plush black - SW 480mm <i>[2173+2154]</i>		1
15	SP1529545-TR35	Backrest Cushion Laguna, Plush black - SW 530mm <i>[2173+2154]</i>		1
16	SP1529538	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 380mm <i>[2173+2153]</i>		1
17	SP1529539	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 430mm <i>[2173+2153]</i>		1
18	SP1529540	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 480mm <i>[2173+2153]</i>		1


Flex 3 / Laguna

Pos.	Part number	Description	Valid from → until	Qty
19	SP1529545	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 530mm <i>[2173+2153]</i>		1

Matrx Posture Back

158885 no

Posture Back

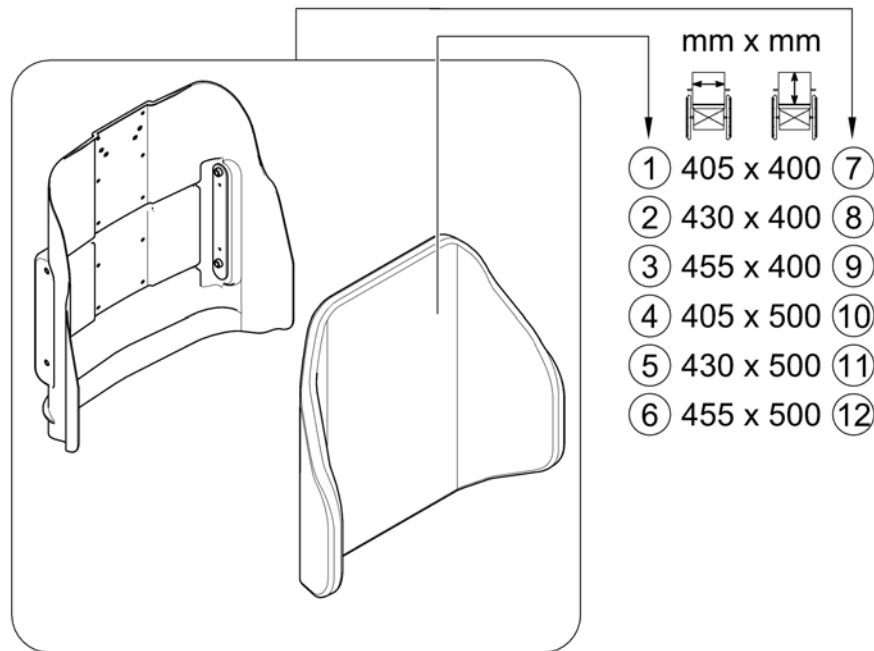


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRS0139-1616	Backrest Cushion - Posture Back 405x400mm (16x16")		1
2	SPTRS0139-1716	Backrest Cushion - Posture Back 430x400mm (17x16")		1
3	SPTRS0139-1816	Backrest Cushion - Posture Back 455x400mm (18x16")		1
4	1544897	Backrest Cushion - Posture Back 405x500mm (16x20")		1
5	SPTRS0139-1720	Backrest Cushion - Posture Back 430x500mm (17x20")		1
6	SPTRS0139-1820	Backrest Cushion - Posture Back 455x500mm (18x20")		1
7	SP1531762	Backrest, Matrx Posture Back 16x16", cpl. <i>[2174]</i>		1
8	SP1531792	Backrest, Matrx Posture Back 17x16", cpl. <i>[2174]</i>		1
9	SP1531822	Backrest, Matrx Posture Back 18x16", cpl. <i>[2174]</i>		1
10	SP1531763	Backrest, Matrx Posture Back 16x20", cpl. <i>[2175]</i>		1
11	SP1531793	Backrest, Matrx Posture Back 17x20", cpl. <i>[2175]</i>		1
12	SP1531823	Backrest, Matrx Posture Back 18x20", cpl. <i>[2175]</i>		1

Matrx Elite T/R Back

1588006 Rev

Elite T/R Back

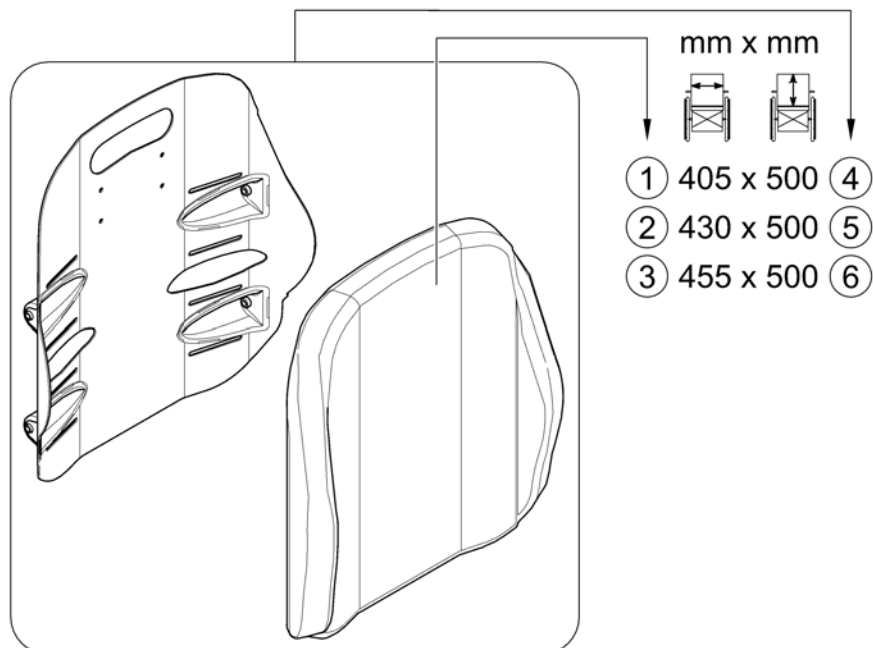


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRS0375-1620	Backrest Cushion - Elite T/R Back 405x500mm (16x20")		1
2	SPTRS0375-1720	Backrest Cushion - Elite T/R Back 430x500mm (17x20")		1
3	SPTRS0375-1820	Backrest Cushion - Elite T/R Back 455x500mm (18x20")		1
4	SP1588270	Backrest, Matrx Elite T/R Back 16x20", cpl. [2177+2121]		1
5	SP1588271	Backrest, Matrx Elite T/R Back 17x20", cpl. [2177+2122]		1
6	SP1588272	Backrest, Matrx Elite T/R Back 18x20", cpl. [2177+2123]		1

Matrx Elite Deep Back

1588713 Rev


Elite Deep Back

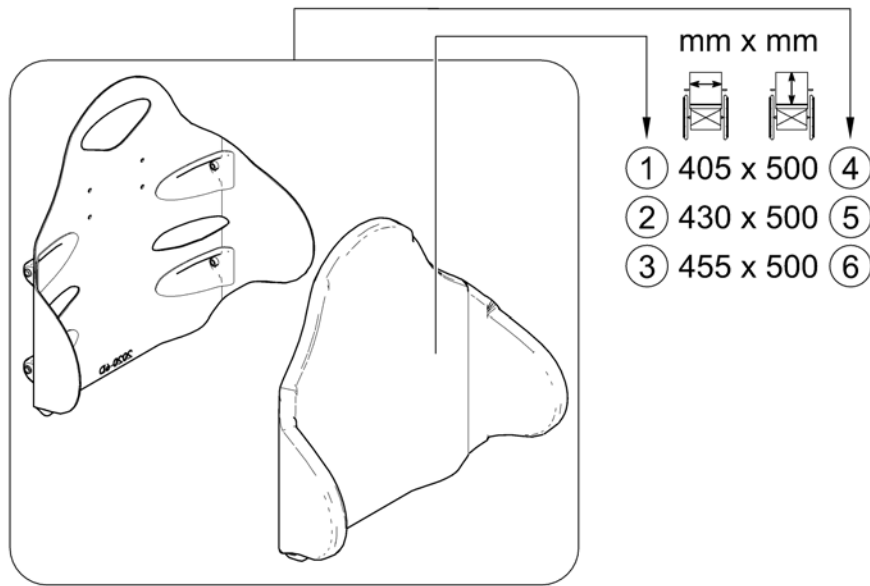

Pos.	Part number	Description	Valid from → until	Qty
1	SPTRS0344-1620	Backrest Cushion - Elite Deep Back 405x500mm (16x20")		1
2	SPTRS0344-1720	Backrest Cushion - Elite Deep Back 430x500mm (17x20")		1
3	SPTRS0344-1820	Backrest Cushion - Elite Deep Back 455x500mm (18x20")		1
4	SP1588273	Backrest, Matrx Elite Deep Back 16x20", cpl. [2178+2121]		1
5	SP1588274	Backrest, Matrx Elite Deep Back 17x20", cpl. [2178+2122]		1
6	SP1588275	Backrest, Matrx Elite Deep Back 18x20", cpl. [2178+2123]		1

Matrx Elite (Full) Back


1588714.00

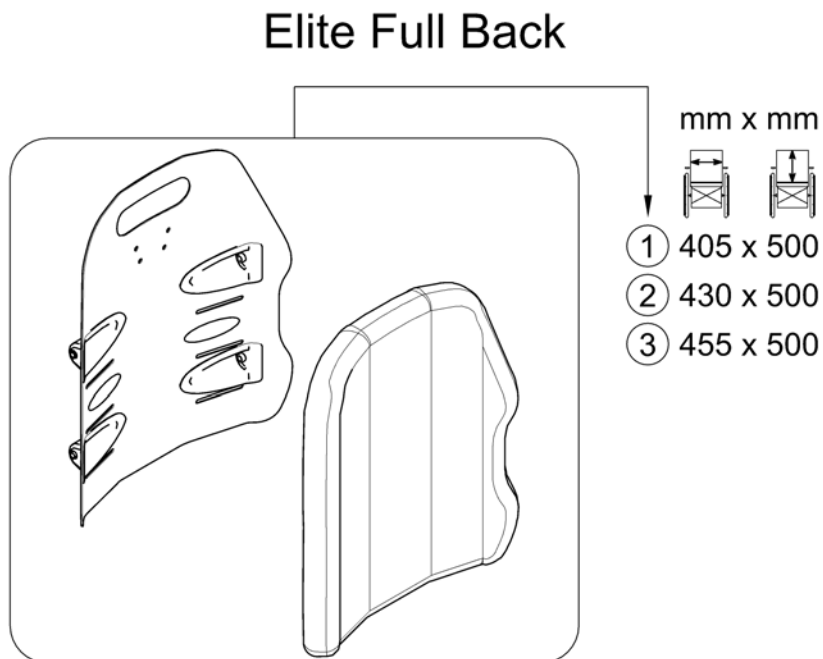

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531824	Backrest, Matrx Elite (Full) Back 16x20", cpl. <i>[2176]</i>		1
2	SP1531831	Backrest, Matrx Elite (Full) Back 17x20", cpl. <i>[2176]</i>		1
3	SP1531832	Backrest, Matrx Elite (Full) Back 18x20", cpl. <i>[2176]</i>		1

Legrest Support, 80° & man. elevating - Standard & Modulite Seat


1441526PAC.kim

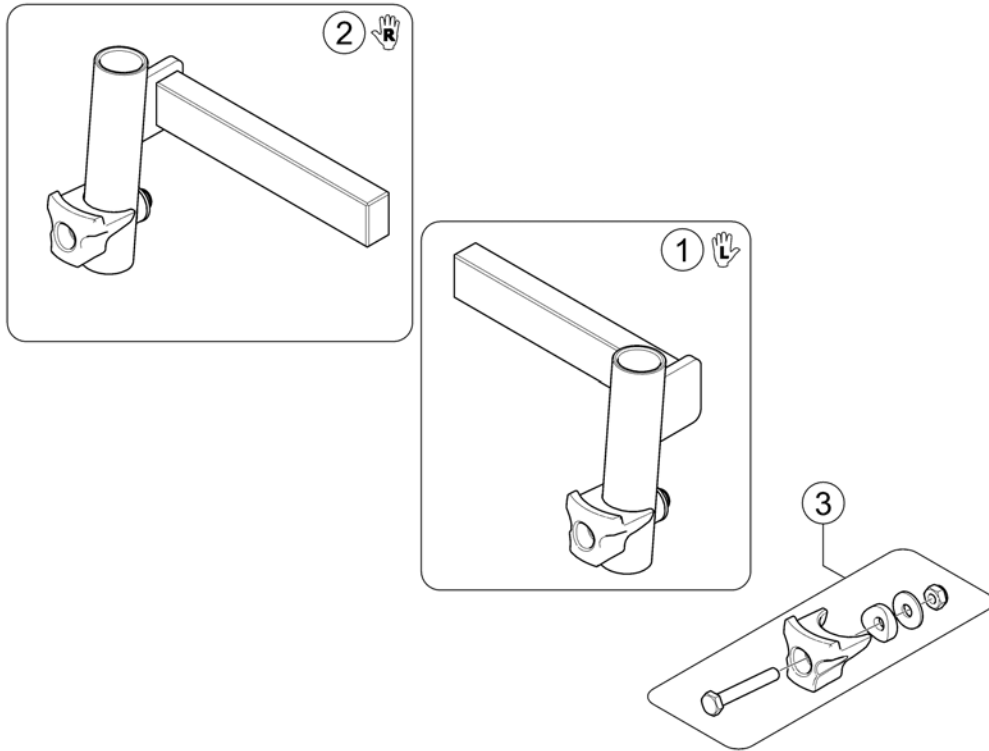

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531195	Legrest support, cpl. - LH <i>We recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>	2015-07-01 →	1
2	SP1531196	Legrest support, cpl. - RH <i>We recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>	2015-07-01 →	1
3	SP1589136	Kit, Support Nose	→ 2015-06-30	1
3	SP1588726	Kit, Support Nose	2015-07-01 →	1

Legrest Support, 80° & man. elevating - Flex-2 & Max Seat


1527089PAC.dwg

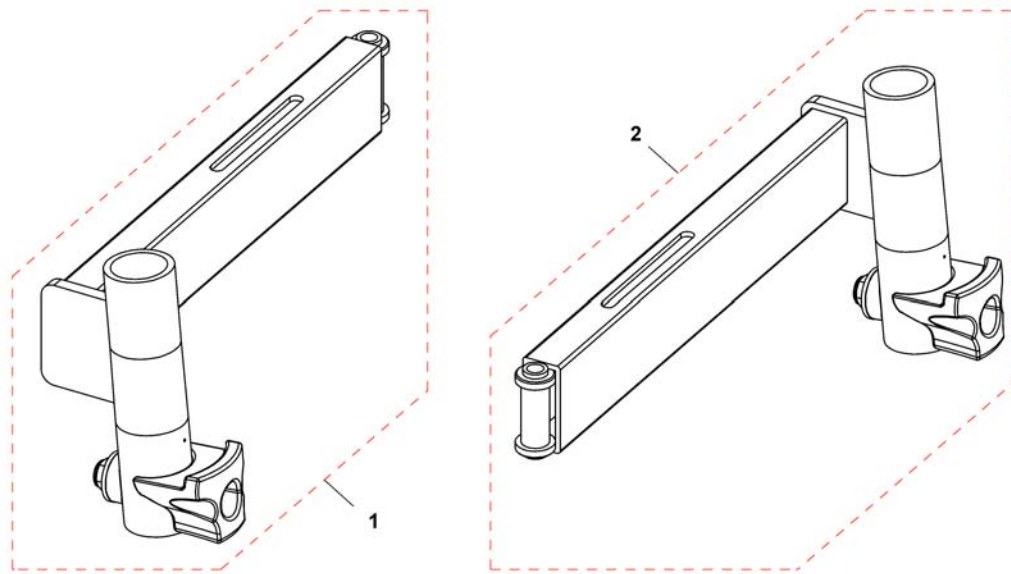

Pos.	Part number	Description	Valid from → until	Qty
1	SP1533502	Legrest support RH <i>we recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>		1
2	SP1533503	Legrest support LH <i>we recommend to exchange the hangers by pairs for products built before 01.01.12, because of new design.</i>		1
-	SP1589136	Kit, Support Nose	→ 2015-06-30	1
-	SP1588726	Kit, Support Nose	2015-07-01 →	1

Legrest Support, Vari series - Standard Seat


1573651.000

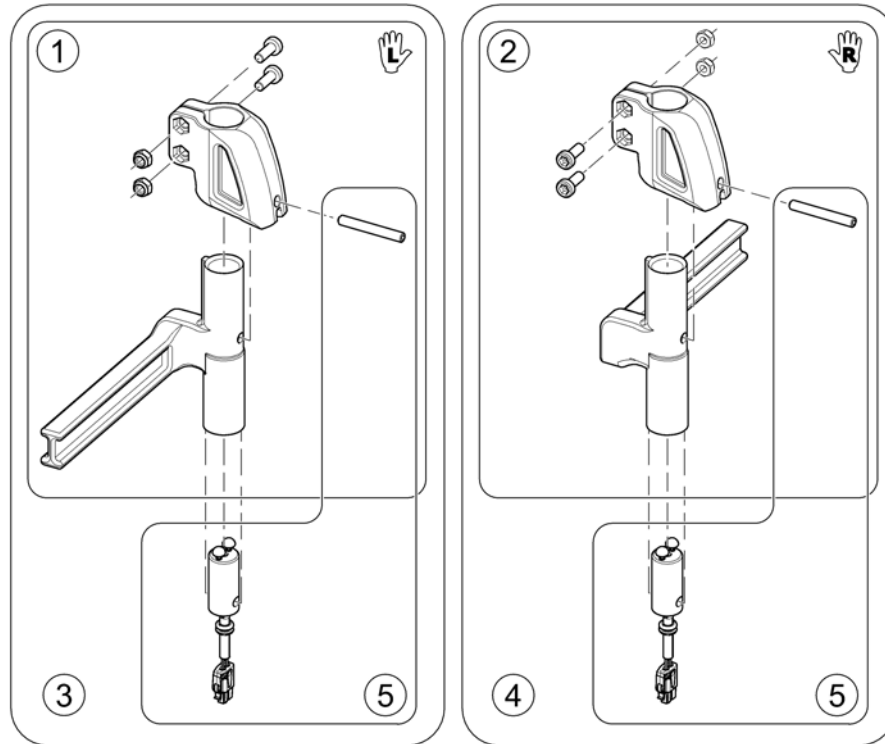

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449733	Legrest Support Vari series, man. 5cm - LH		1
2	SP1449732	Legrest Support Vari series, man. 5cm - RH		1
3	SP1551324	Legrest Support Vari series, el. 5cm - LH		1
4	SP1551323	Legrest Support Vari series, el. 5cm - RH		1
5	SP1450232	Power Supply Cable Legrest (550mm)		1
-	SP1524744	Adaptercable Actuator AMP -> DX2 <i>with Lifter</i>		1
-	SP1547305	Extension cable for actuator <i>without Lifter</i>		1

Legrest Support, Vari series - Flex-2 & Max Seat


1573651.000

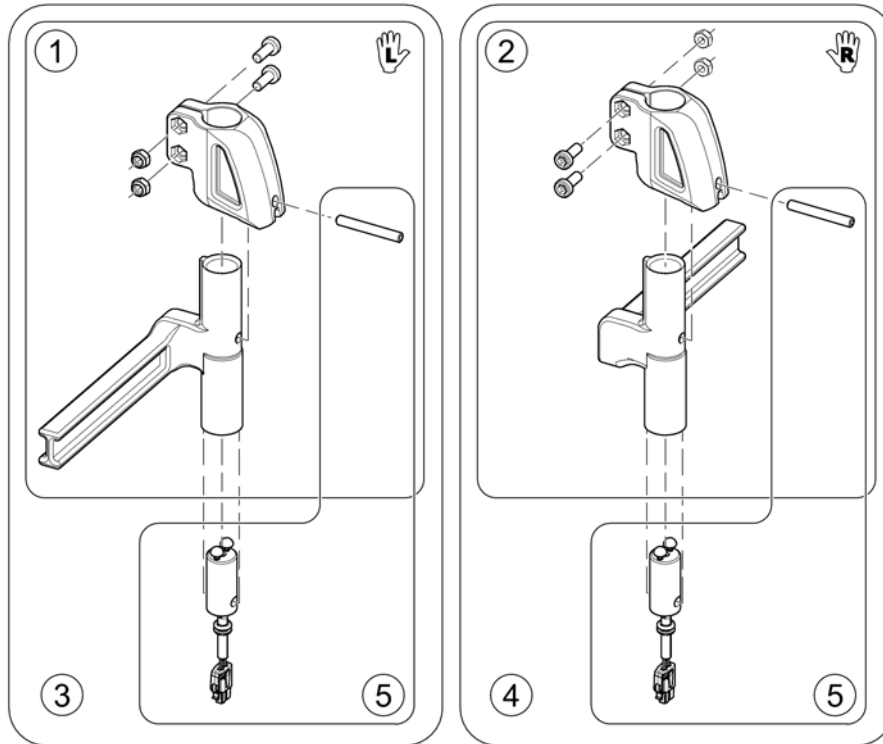

Pos.	Part number	Description	Valid from → until	Qty
1	SP1526909	Footrest/Legrest receptacle, manuell, LH, 5cm		1
2	SP1526910	Footrest/Legrest receptacle, manuell, RH, 5cm		1
2	SP1562029	Legrest support, powered, RH <i>Only Max Seat!</i>		1
3	SP1551326	Legrest support, powered, LH		1
4	SP1551325	Legrest support, powered, RH		1
5	SP1450232	Power Supply Cable Legrest (550mm)		1
5	SP1450228	Power Supply Cable Legrest (990mm) <i>only Max Seat RH!</i>		1
-	SP1524744	Adaptercable Actuator AMP -> DX2 <i>with Lifter</i>		1
-	SP1547305	Extension cable for actuator <i>without Lifter</i>		1

Legrest Support, Vari series - Modulite Seat


1573651.000

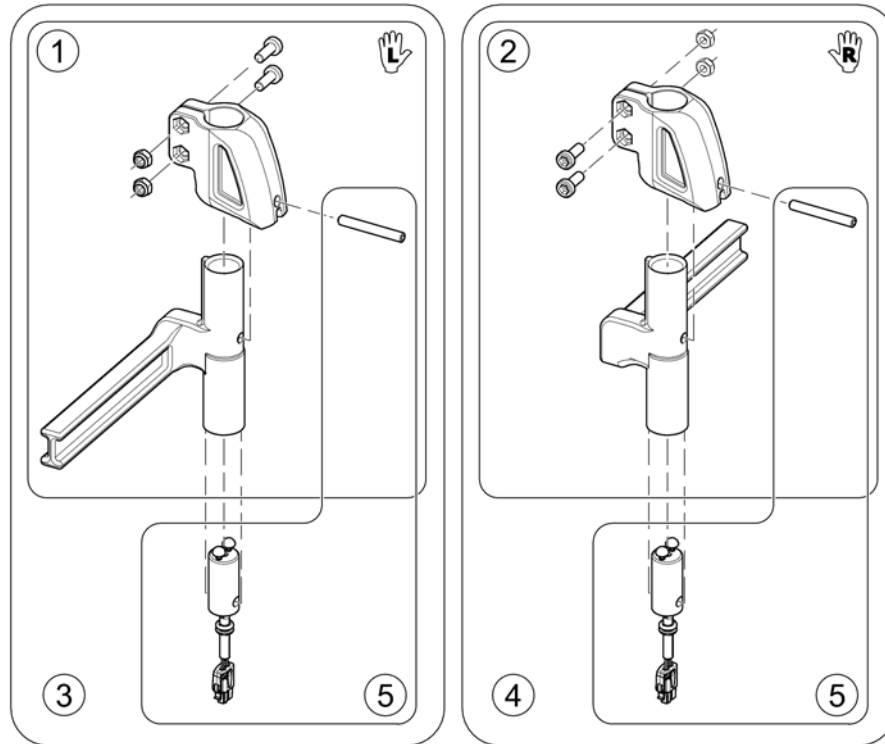

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449733	Legrest Support Vari series, man. 5cm - LH		1
2	SP1449732	Legrest Support Vari series, man. 5cm - RH		1
3	SP1529951	Footrest/Legrest receptacle, powered, LH, 5cm		1
4	SP1529949	Footrest/Legrest receptacle, powered, RH, 5cm		1
5	SP1527775	Power Supply Cable Legrest (650mm)		1

Hanger, Std. 80° (01.07.15 =>)


1588852.kim

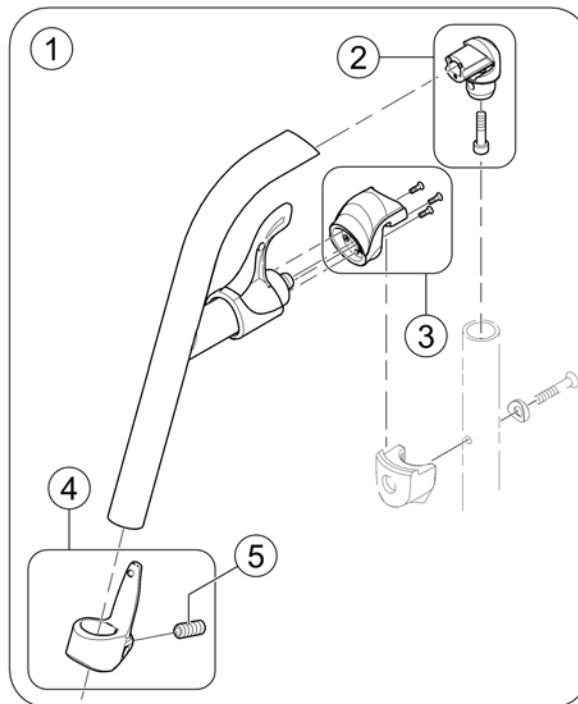

Pos.	Part number	Description	Valid from → until	Qty
1	SP1574349-9005	Hanger Standard 80° cpl. - black		1
2	SP1588727	Kit, Pivot Std. 80° Hanger		1
3	SP1588731	Nose locking Kit		1
4	SP1588730	Calf Strap Holder, cpl.		1
5	1424571	Springscrew		1

Invacare® Kite | Legrests / Hangers
Hanger, Std. 80° (=> 30.06.15)


1493054PAC.kim

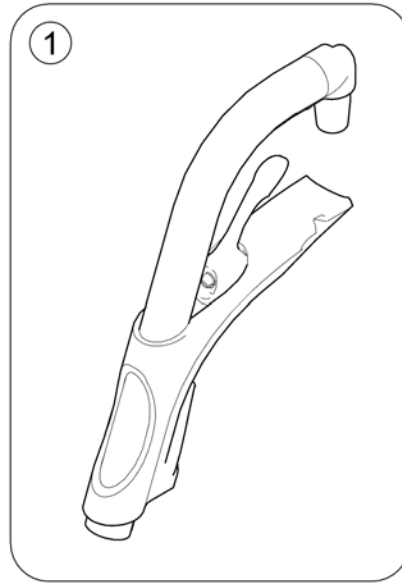

Pos.	Part number	Description	Valid from → until	Qty
1	SP1493055	Hanger 80°, Fixed		1

Hanger, manually height adjustable


1513006.dwg

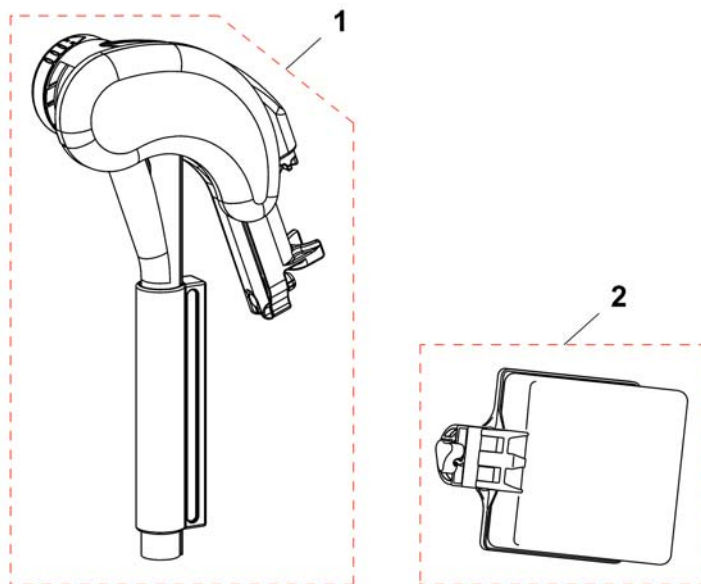

Pos.	Part number	Description	Valid from → until	Qty
1	1513006	HANGER ALU2008, AA, RIGHT, CPL		1
1	1513007	HANGER ALU2008, AA, LEFT, CPL		1
2	1516314	CALF PAD 15 (165x170x80), CPL		1
2	1518532	Calfpad, 19", Dartex black, cpl.		1


Hanger Vari F

144934 / X.dwg

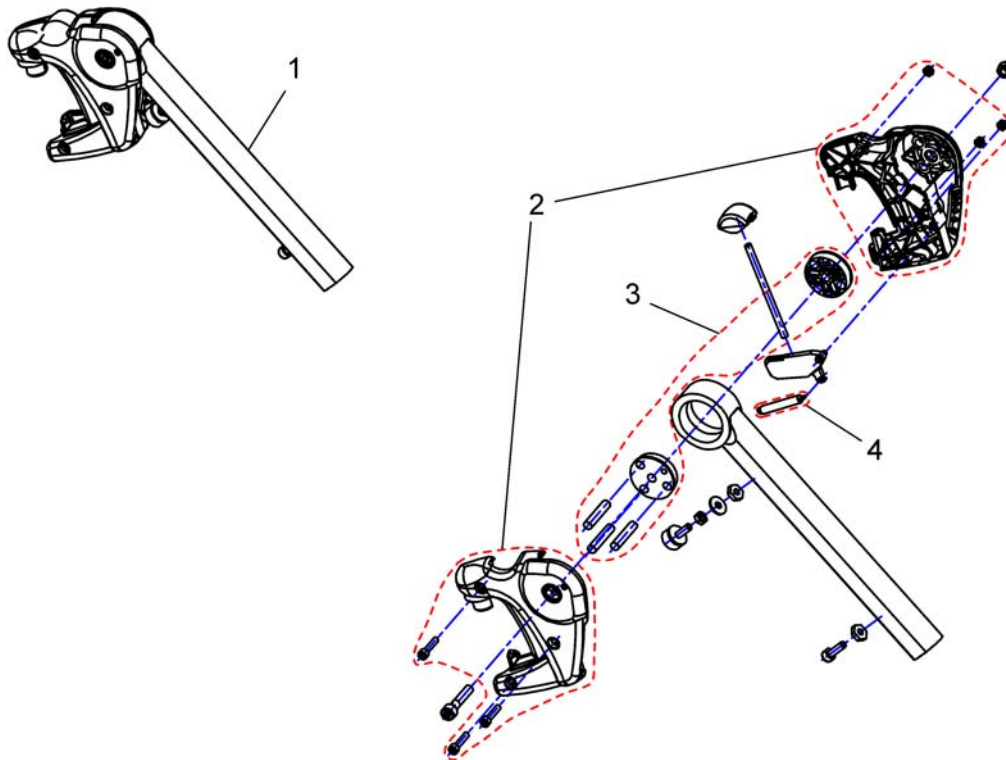

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449347	Hanger Vari F LH cpl.		1
1	SP1449346	Hanger Vari F RH cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	1434956	Mechanic		1
4	SPZ173178	Spring		1


Hanger Vari A (04/2009 =>)

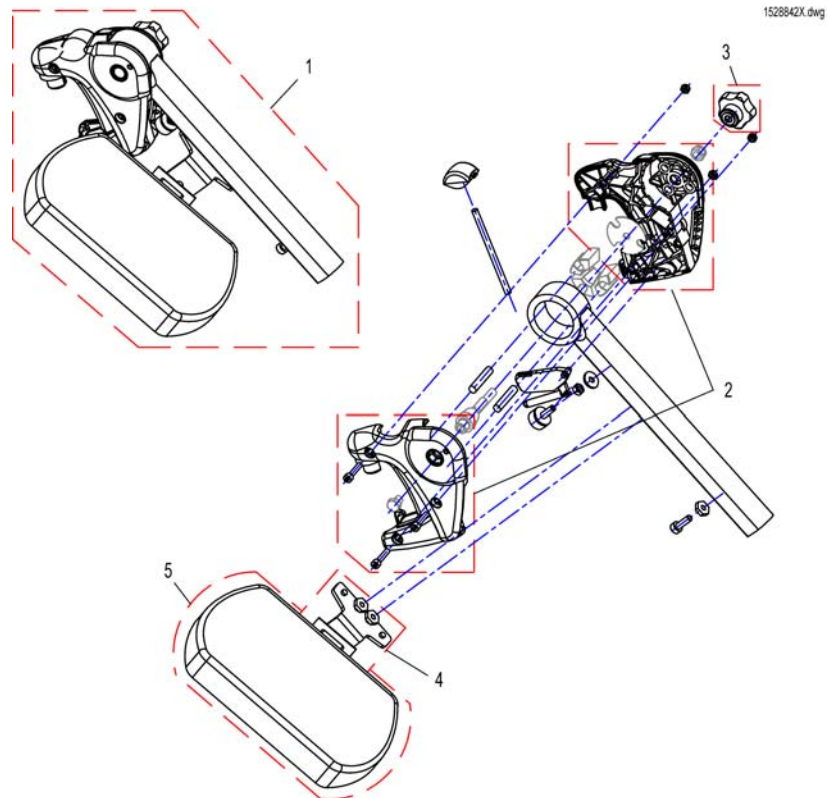

Pos.	Part number	Description	Valid from → until	Qty
-	SP1527820	Rework-Kit Vari A Legrest Version 1 => Version 2 <i>Vari A Version 1 => Version 2 Upgrade Kit</i>		1
1	SP1527498	Hanger Vari A LH, cpl.		1
1	SP1527492	Hanger Vari A RH, cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	L34011	Handknob M8 Tristarfemale		1
4	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
4	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
5	6301333	Legrest calf pad		1

Hanger Vari ADE/ADP electric (07/2005 =>)


1449407X.dwg

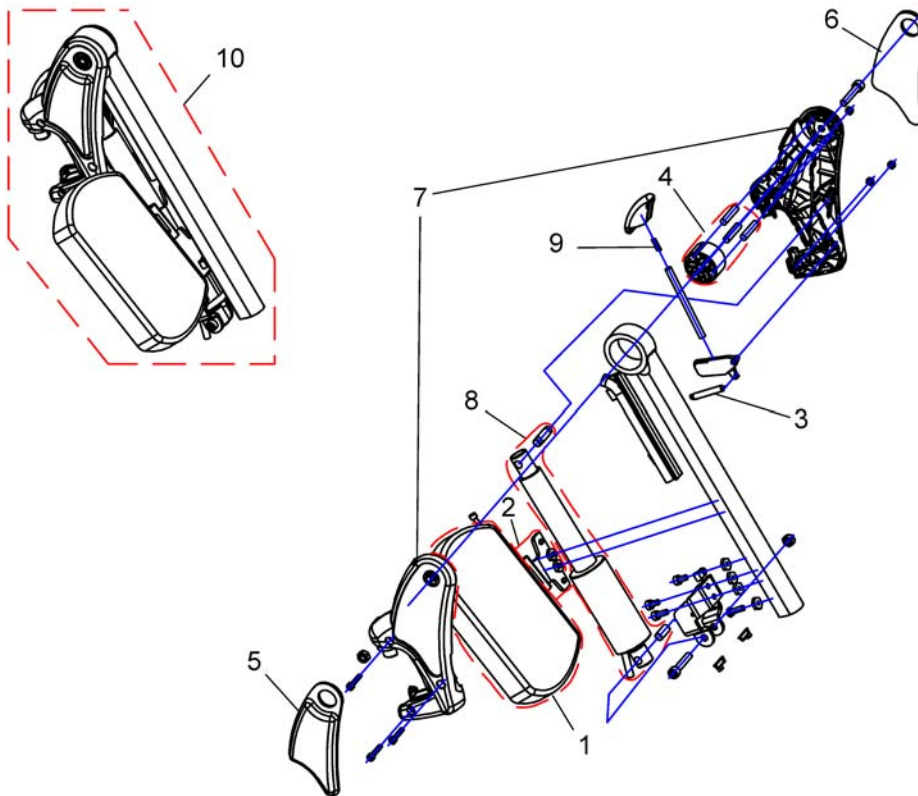

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450448	Knee Upholstery RH		1
6	SP1450410	Knee Upholstery LH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ151006	Actuator with circuit board, preassembled		1
9	SPN12196	Spring		1
10	SP1449408	Hanger ADP RH cpl.		1
10	SP1449407	Hanger ADP LH cpl.		1

Footrests Vari series


1423156X.png

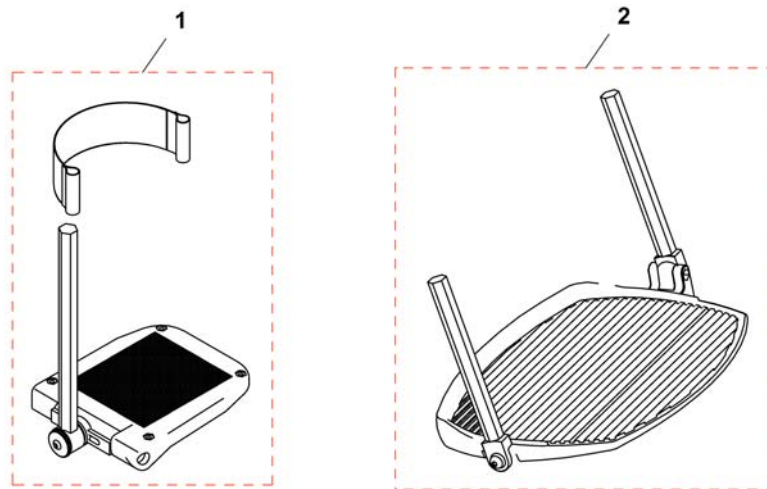

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1
1	SP1419345	Footplate RH Anlge And Depth Adjustable, With Support Tube cpl.		1
2	SP1423156	Footboard, pre-assembled		1


Footplate, Standard

158854.iss

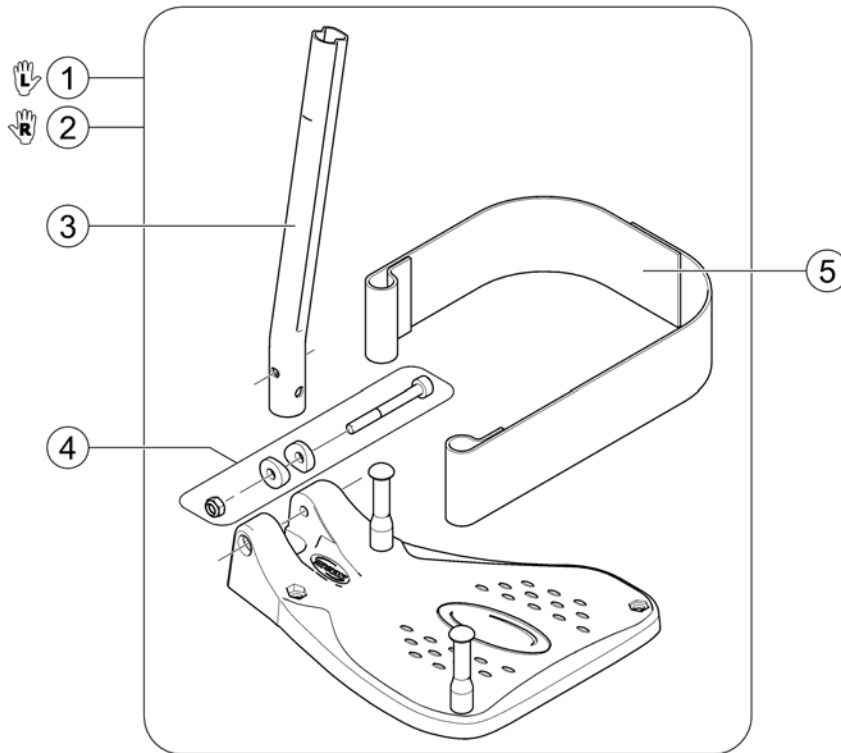

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537944	Footrest Tube with Footplate for Hanger, cpl.43-45,5cm LH		1
2	SP1537942	Footrest Tube with Footplate for Hanger, cpl.43-45,5cm RH		1
3	1434561-0010	Footrest Tube, Short		1
4	1430433	Screw Kit, Footrest Fixed		1
5	SP1442467	Heel Strap cpl.		1


Footrests 80° & man. elevating

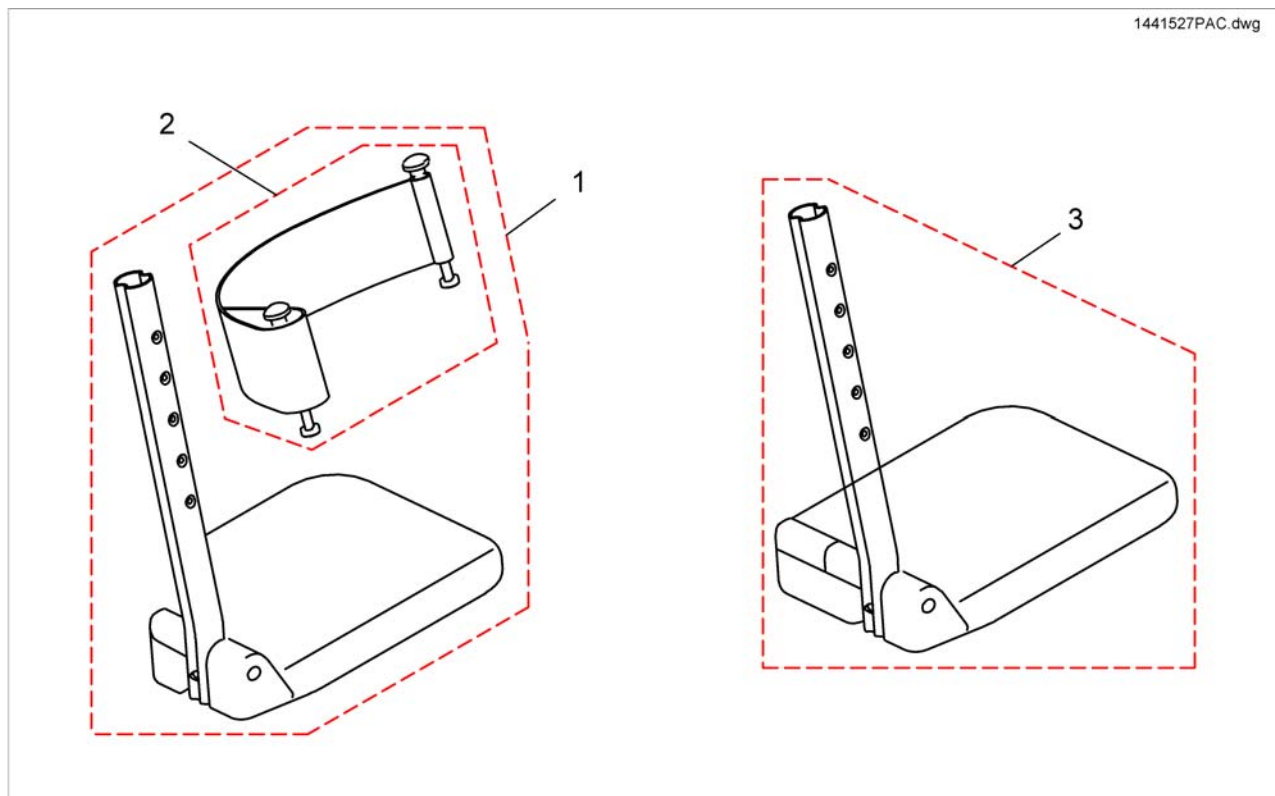

Pos.	Part number	Description	Valid from → until	Qty
1	SP1442851	Lower Legrest Standard With Heel Strap, Width 43-45,5cm, Right		1
1	SP1442852	Lower Legrest Standard With Heel Strap, Width 43-45,5cm, Left		1
2	1430402	HEELSTRAP COMPLETE		1
3	1428367	FOOTREST 24ALU2003,LOWER PART,FIXED,LONG,LEFT		1
3	1428368	FOOTREST 24ALU2003,LOWER PART,FIXED,LONG,RIGHT		1

Footplate, depth-, angledadjustable, 80°


1428654X.iso

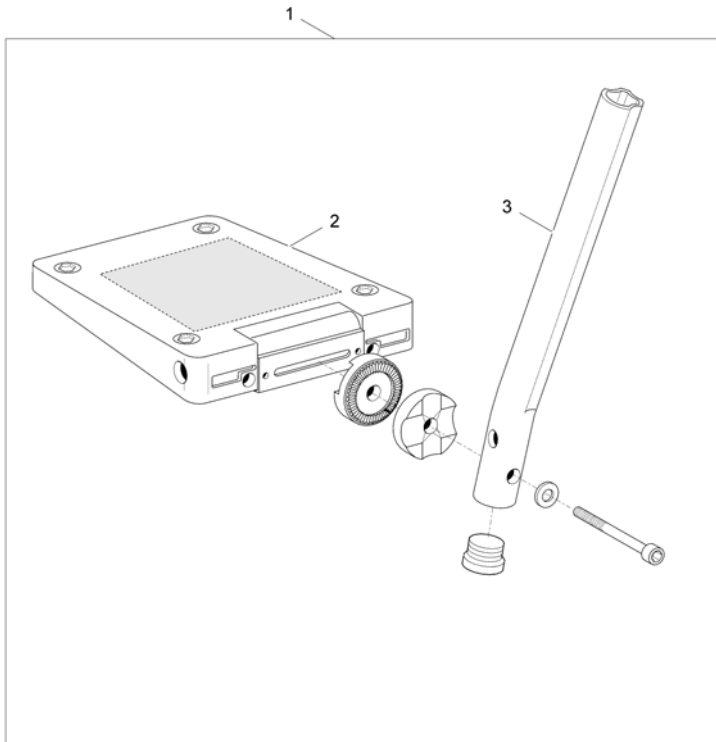

Pos.	Part number	Description	Valid from → until	Qty
1	1428660	Footrest a&d adjustable, Lh, SW455mm, standard tube, swing away frame		1
1	1428654	Footrest a&d adjustable, Rh, SW455mm, standard tube, swing away frame		1
2	5106594	Footplate 17 aa, Cpl.		1
3	1434561-0010	Footrest Tube, Short		1


Footrests Vari, Flip-Up (01.08.10 =>)

152964X.jpg

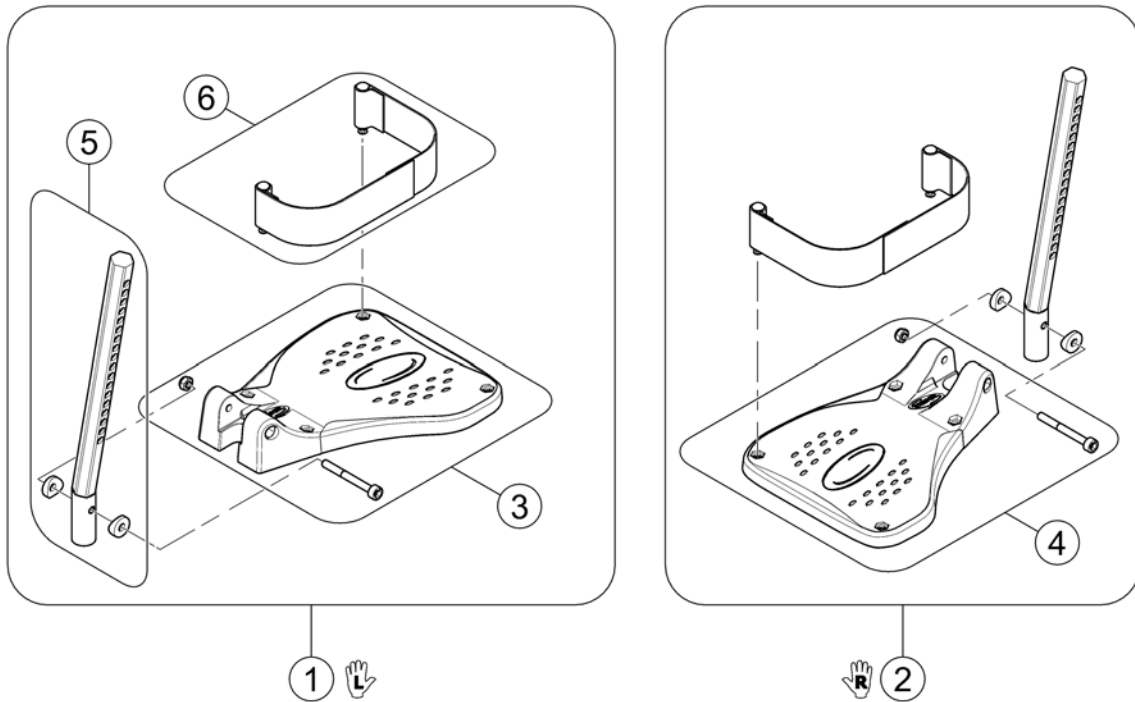


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541575	Footplate flip up, LH, cpl.		1
2	SP1541574	Footplate flip up, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1543045	Plug-in tube LH/RH		1
6	SP1442467	Heel Strap cpl.		1

Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)

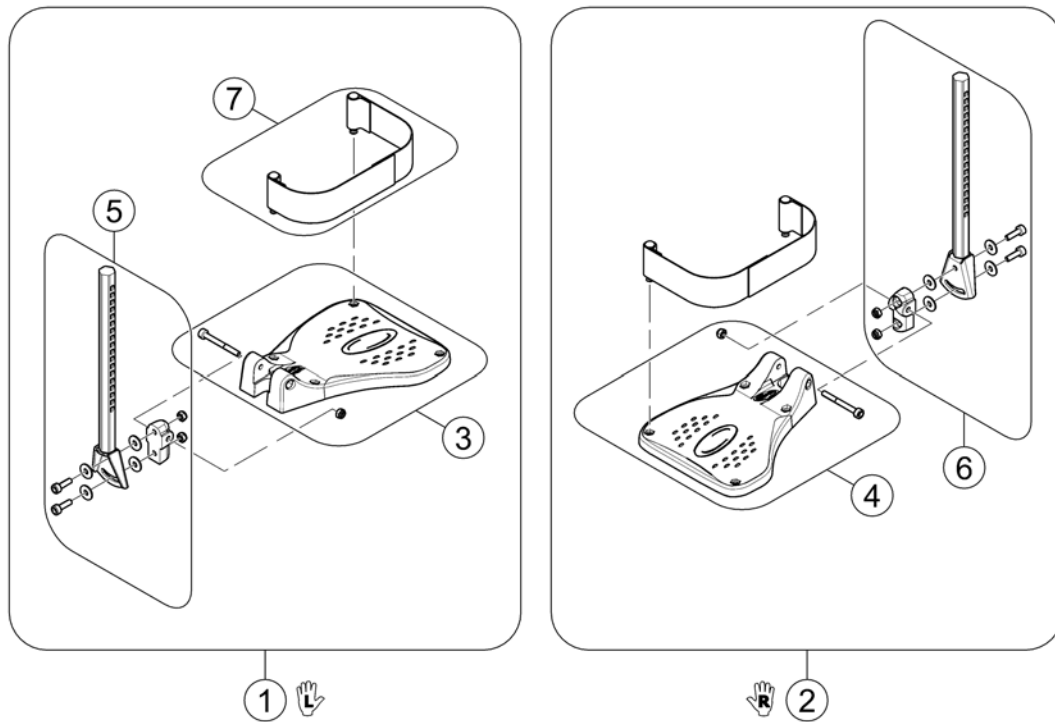

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541580	Footplate flip up, angle adjustable, LH, cpl.		1
2	SP1541579	Footplate flip up, angle adjustable, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1423204	FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.		1
6	SP1423203	FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.		1
7	SP1442467	Heel Strap cpl.		1

Footboard

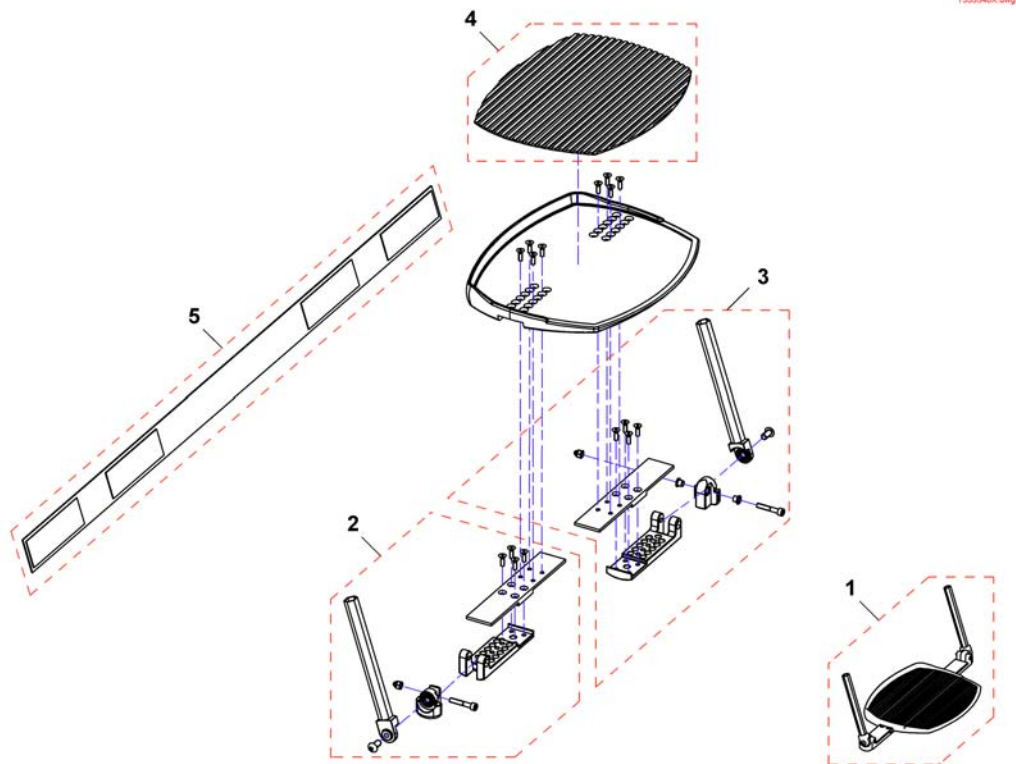

1533540X.dwg

Pos.	Part number	Description	Valid from → until	Qty
1	SP1533540	Footboard for Seatwidth 53		1
2	SP1552599	Footboard bracket RH		1
3	SP1552600	Footboard bracket LH		1
4	Z174159	Rubber mat		1
5	SP1552767	Calf strap		1

Armrests


SPOKTE20.iss



Pos.	Part number	Description	Valid from → until	Qty
1	-	Armrest for Standard Seat		1
2	-	Armrest for Flex2 Seat		1
3	-	Armrest for Max Seat		1
4	-	Armrests for Modulite Seat		1


Armrest for Standard Seat

1571952.iss

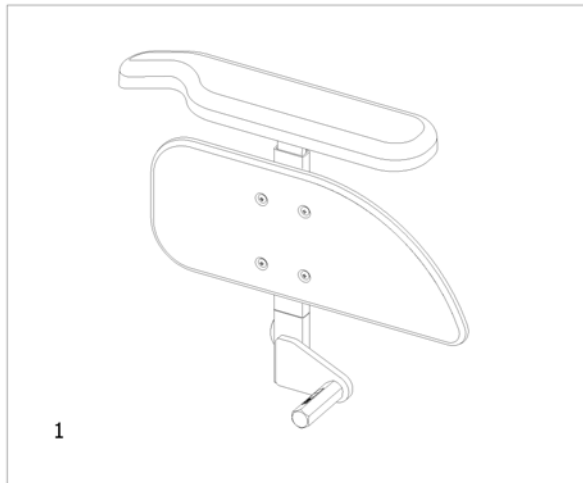

Pos.	Part number	Description	Valid from → until	Qty
1	-	T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==> [0606+0647/0649]		1


T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>

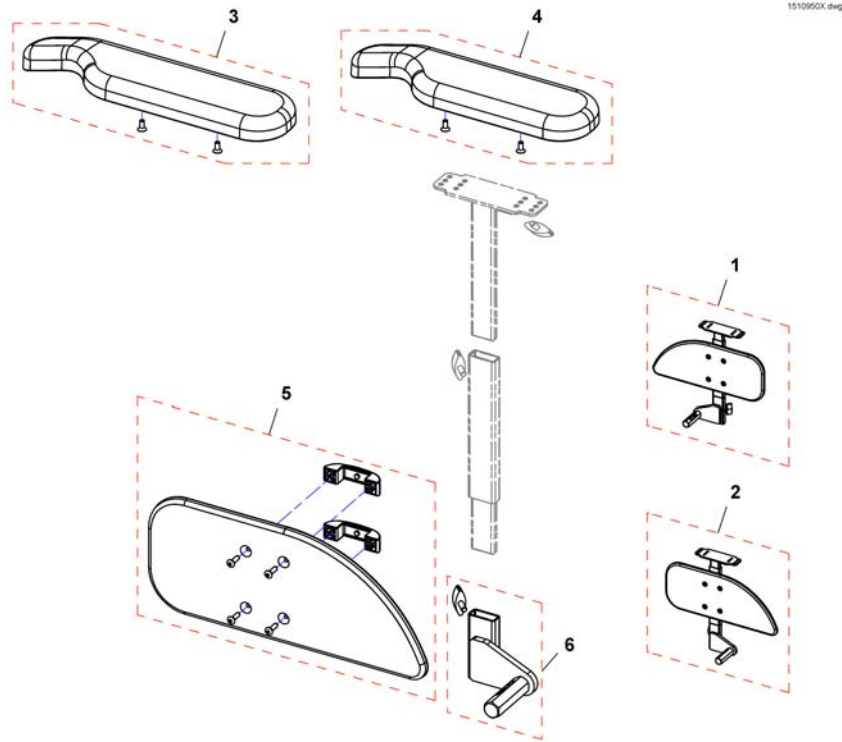

Pos.	Part number	Description	Valid from → until	Qty
1	SP1510935	Armrest, without armpad, RH		1
2	SP1510950	Armrest, without armpad, LH		1
3	SP1528846	Armpad, Extended (375mm), LH		1
3	SP1528845	Armpad, Extended (375mm), RH		1
4	SP1443004	Armpad, Standard (325mm), LH		1
4	SP1449043	Armpad, Standard (325mm), RH		1
5	SP1541505	Skirt guard, RH		1
5	SP1541506	Skirt guard, LH		1
6	SP1541510	Bracket for T-arm, RH		1
6	SP1541511	Bracket for T-arm, LH		1

Armrest for Flex2 Seat


1571915.iso

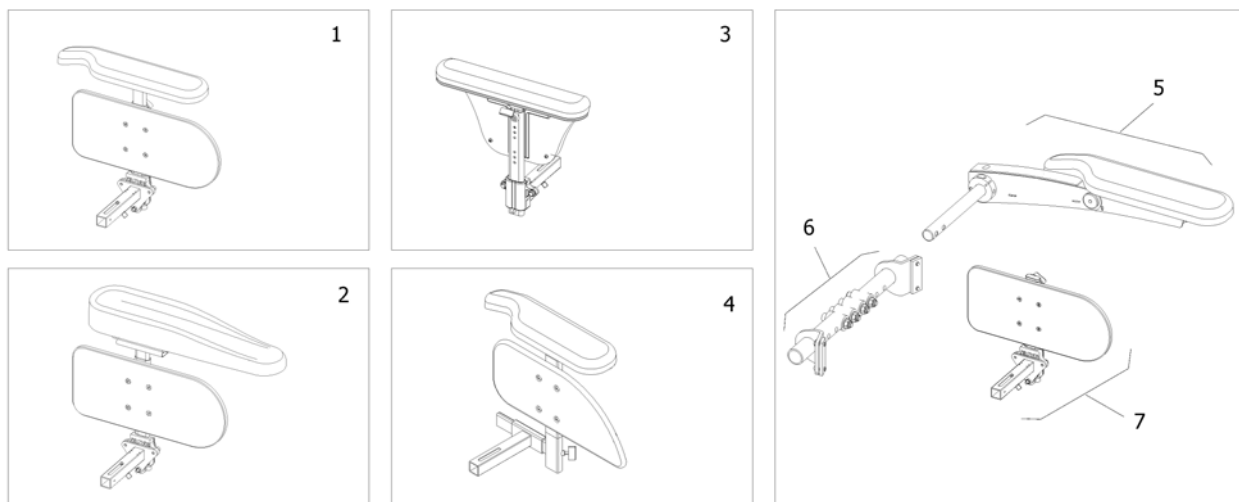

Pos.	Part number	Description	Valid from → until	Qty
1	-	Armrest long [0670]		1
2	-	Armrest Hemiplegic [0670+0671/0672]		1
3	-	Armrest Azalea style [0625]		1
4	-	T-Armrest detachable [0606]		1
5	-	Flip-up armrest [0621]		1
6	-	Mounting bracket Flip-up armrests [0621]		1
7	-	Skirt guard Flip-up armrest [0623]		1


Armrest long

1549087X.dwg

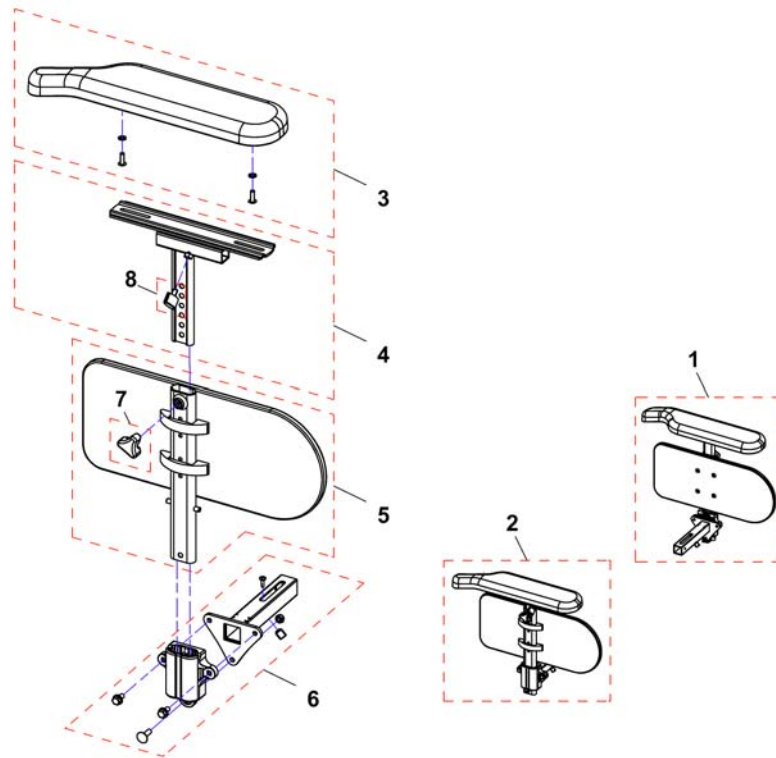

Pos.	Part number	Description	Valid from → until	Qty
1	SP1549088	Long armrest, height adjustable, LH, cpl.		1
2	SP1549087	Long armrest, height adjustable, RH, cpl.		1
3	SP1528846	Armpad, Extended (375mm), LH		1
3	SP1528845	Armpad, Extended (375mm), RH		1
4	SP1433317	T-Arm, LH		1
4	SP1433316	T-Arm, RH		1
5	1415521	Skirtguard LH, cpl.		1
5	1415520	Skirtguard RH, cpl.		1
6	SP1528816	Armrest Bracket RH/LH		1
7	1434623	Ball Screw Court 75mm		1
8	L31001	Wing Screw M6X10		1

Armrest Hemiplegic

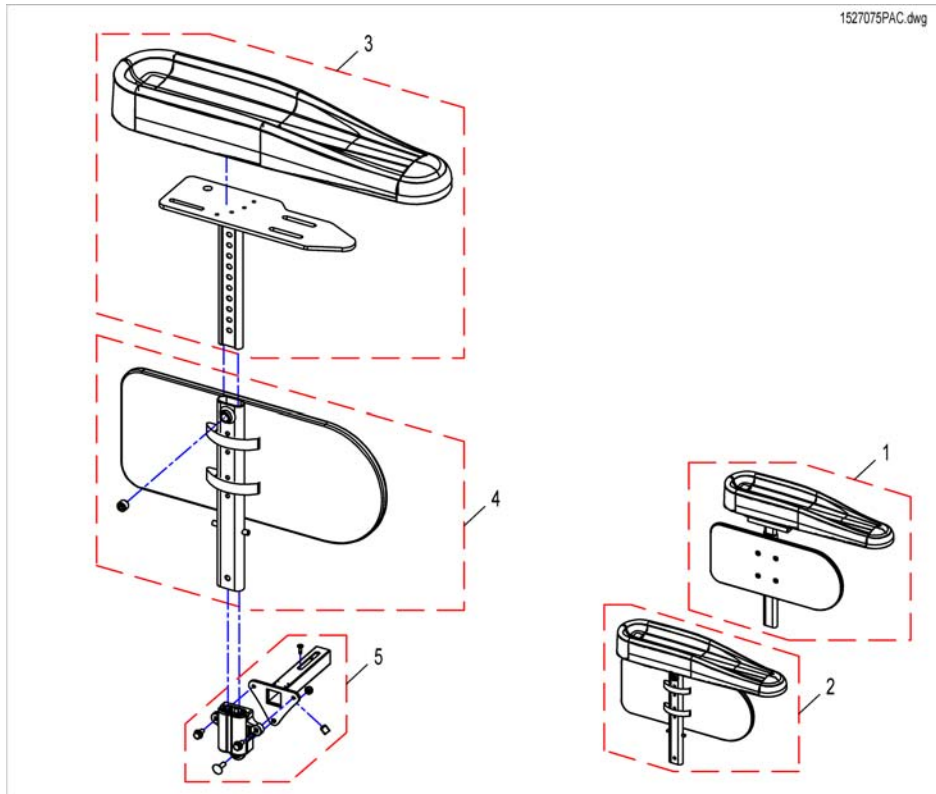

Pos.	Part number	Description	Valid from → until	Qty
1	SP1527076	Armrest Hemiplegic LH		1
2	SP1527075	Armrest Hemiplegic RH		1
3	1493219	Rh hemi dartex armpad complete		1
3	1493220	Lh hemi dartex armpad complete		1
4	1415520	Skirtguard RH, cpl.		1
4	1415521	Skirtguard LH, cpl.		1
5	SP1528816	Armrest Bracket RH/LH		1

Armrest Azalea style


1527196PAC.dwg




Pos.	Part number	Description	Valid from → until	Qty
1	SP1527196	Armrest RH/LH, cpl.		1
2	L31001	Wing Screw M6X10		1
3	SP1528816	Armrest Bracket RH/LH		1

T-Armrest detachable

1559709.iso

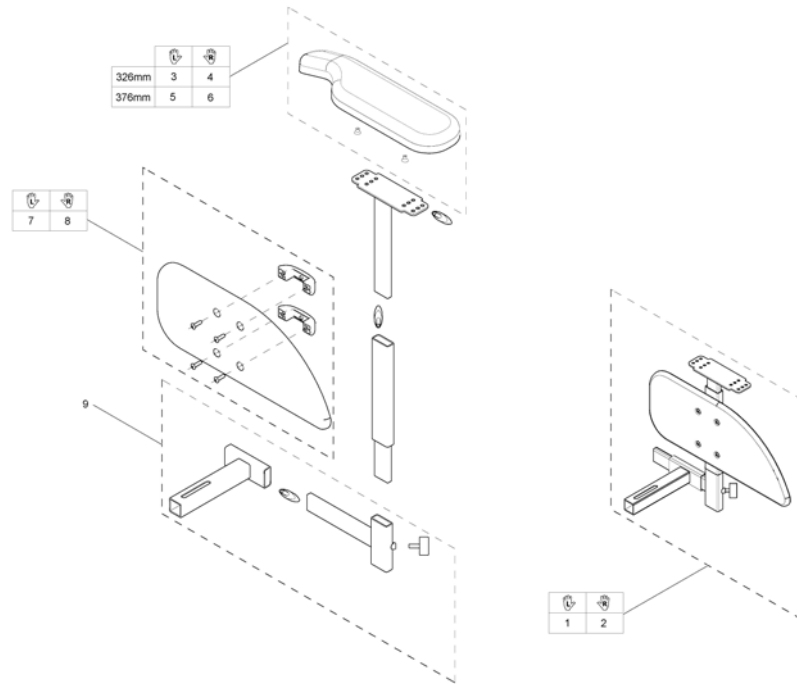


Pos.	Part number	Description	Valid from → until	Qty
1	SP1556920	Armrest LH, without Armpad		1
2	SP1556925	Armrest RH, without Armpad		1
3	SP1443004	Armpad, Standard (325mm), LH		1
4	SP1449043	Armpad, Standard (325mm), RH		1
5	SP1528846	Armpad, Extended (375mm), LH		1
6	SP1528845	Armpad, Extended (375mm), RH		1
7	SP1541506	Skirt guard, LH		1
8	SP1541505	Skirt guard, RH		1
9	SP1559710	Receiver for armrest		1

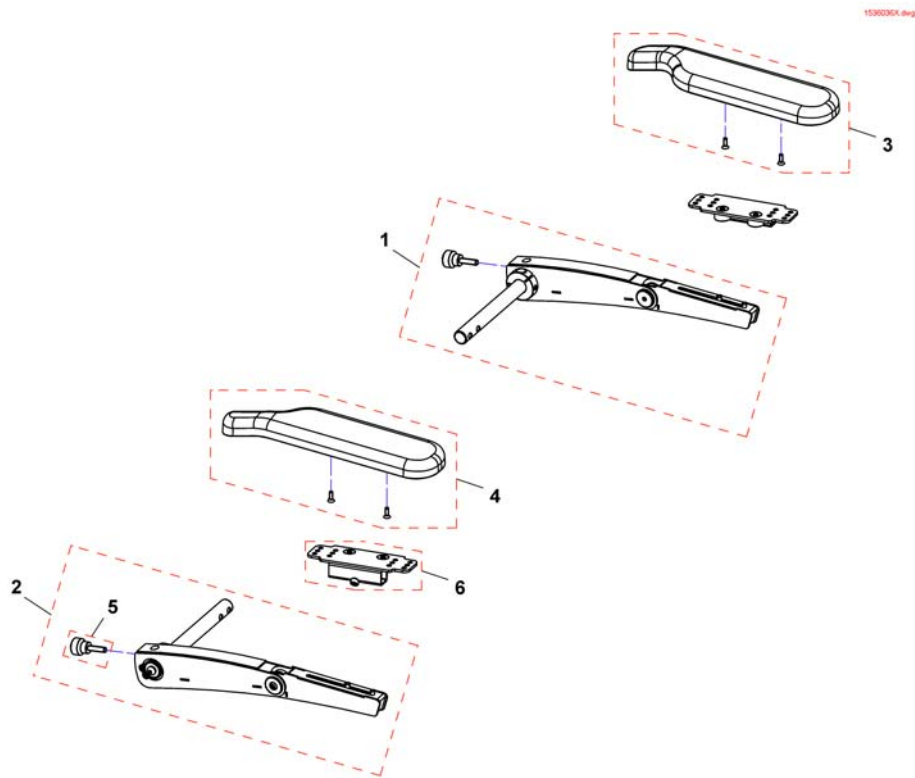

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536100	Following armrest, long, LH, cpl.		1
2	SP1536099	Following armrest, long, RH, cpl.		1
3	SP1528846	Armpad, Extended (375mm), LH		1
4	SP1528845	Armpad, Extended (375mm), RH		1
5	SP1535504	Knurled screw M8x30		1
6	SP1535244	Armpad holder		1

Mounting bracket Flip-up armrests


1548568.dwg

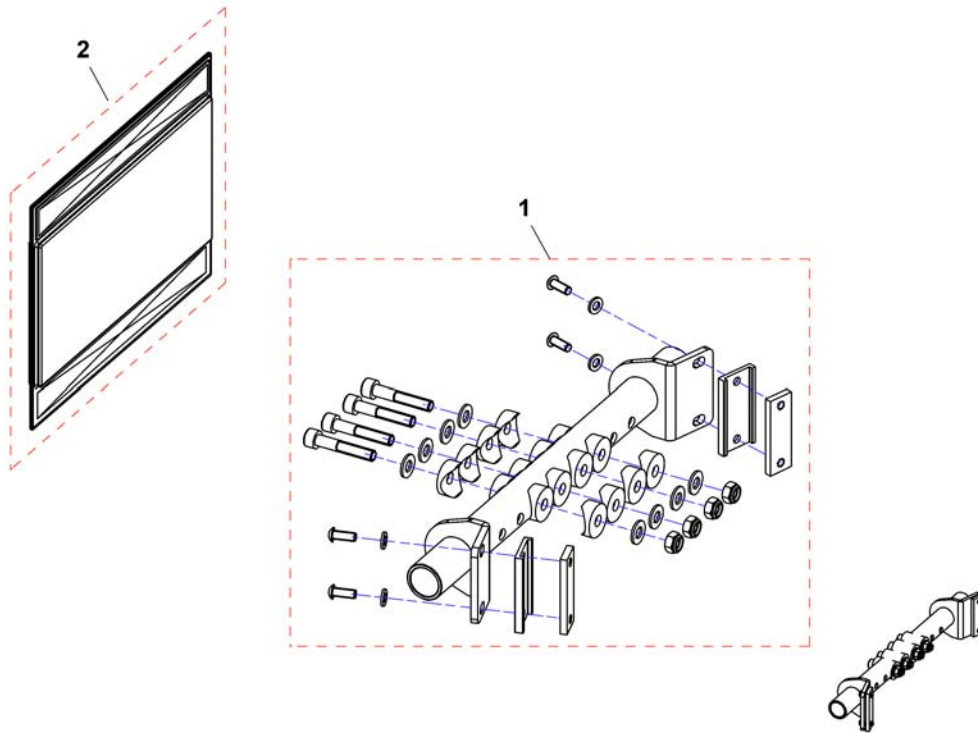

Pos.	Part number	Description	Valid from → until	Qty
1	SP1548568	Bracket for flip-up armrest, SW 39		1
1	SP1548569	Bracket for flip-up armrest, SW 44		1
1	SP1548631	Bracket for flip-up armrest, SW 49		1
2	SP1550410	Protection cover		1

Skirt guard Flip-up armrest


1549094.dwg

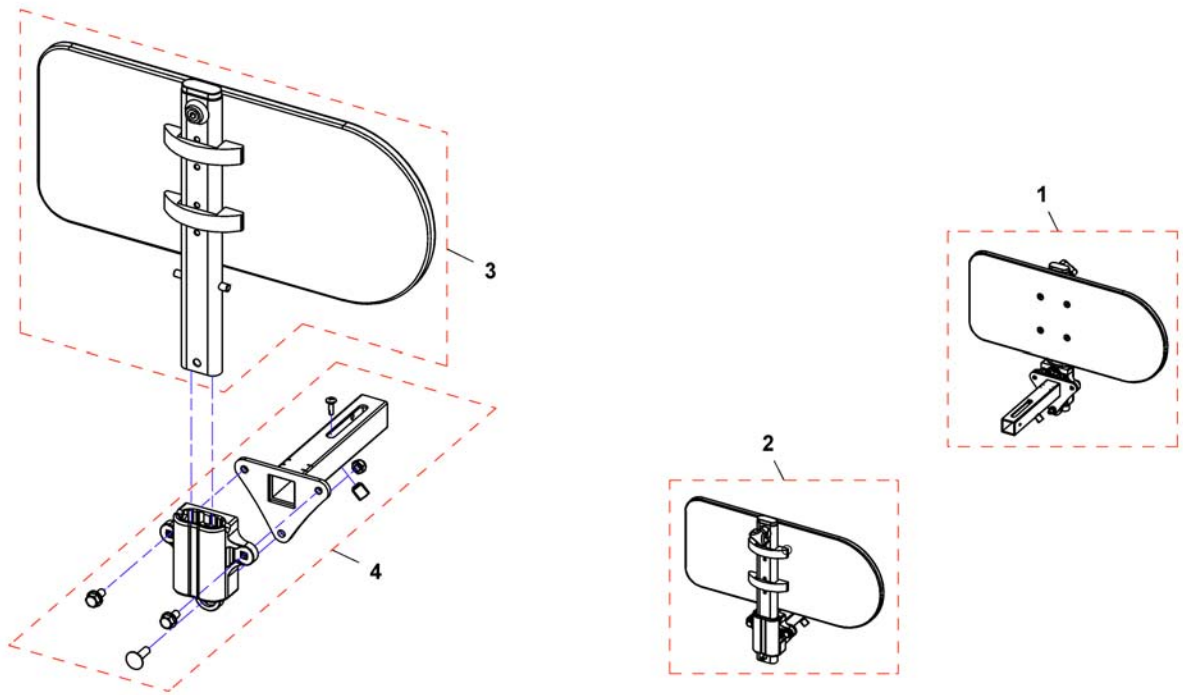

Pos.	Part number	Description	Valid from → until	Qty
1	SP1549095	Skirt guard, LH		1
2	SP1549094	Skirt guard, RH		1
3	1415521	Skirtguard LH, cpl.		1
3	1415520	Skirtguard RH, cpl.		1
4	SP1528816	Armrest Bracket RH/LH		1

Armrest for Max Seat


1571916.iso

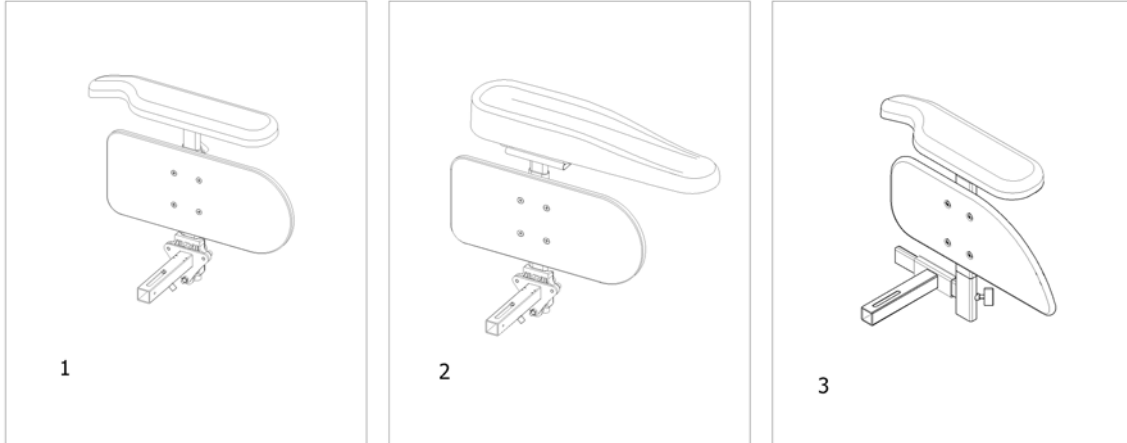

Pos.	Part number	Description	Valid from → until	Qty
1	-	Armrest long [0670+0649]		1
2	-	Armrest Hemiplegic [0670+0671/0672]		1
3	-	T-Armrest detachable [0606+0649]		1

Armrest long


1552491X.dwg

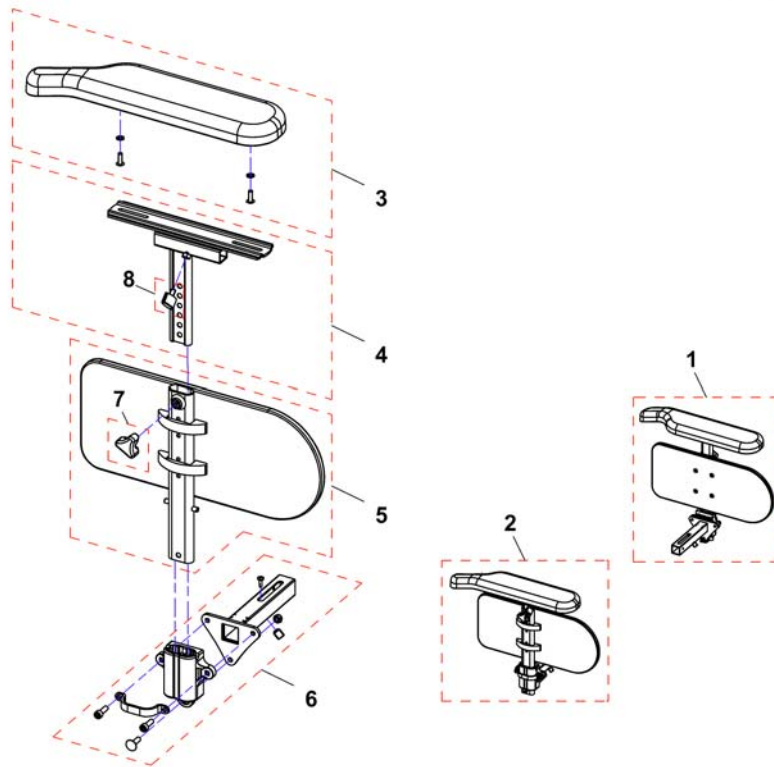

Pos.	Part number	Description	Valid from → until	Qty
1	SP1552491	Armrest compl. LH		1
2	SP1552492	Armrest compl. RH		1
3	SP1528846	Armpad, Extended (375mm), LH		1
3	SP1528845	Armpad, Extended (375mm), RH		1
4	SP1433317	T-Arm, LH		1
4	SP1433316	T-Arm, RH		1
5	1415521	Skirtguard LH, cpl.		1
5	1415520	Skirtguard RH, cpl.		1
6	SP1525035	Reinforced bracket HD armrest RH/LH		1

Armrests for Modulite Seat


1571738.000

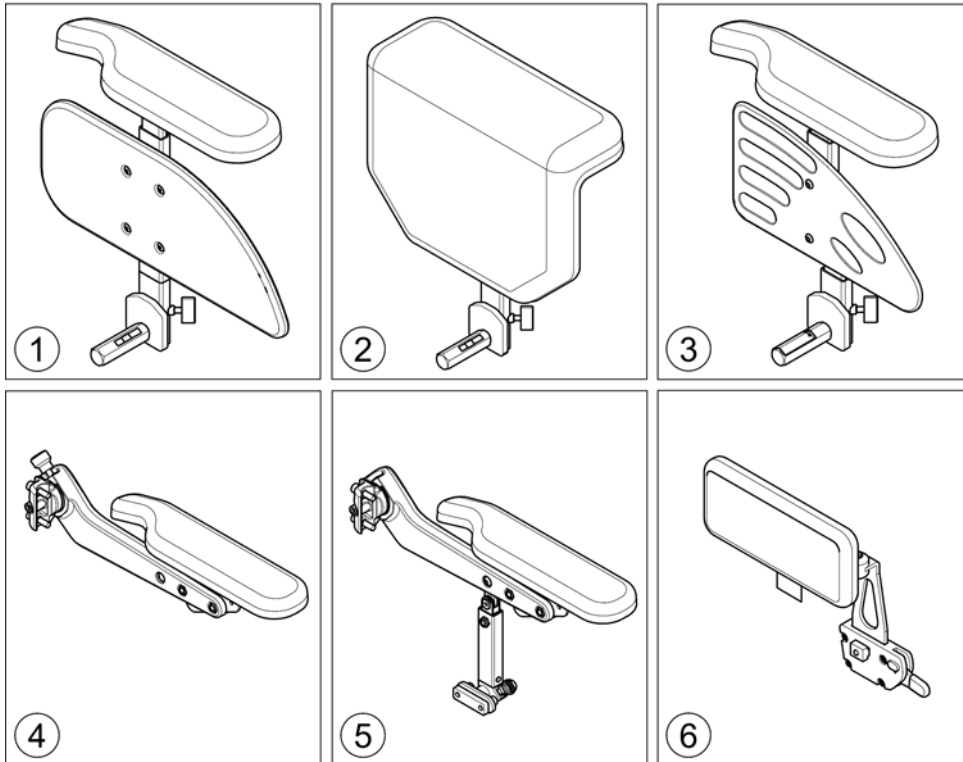

Pos.	Part number	Description	Valid from → until	Qty
1	-	T-Armrest [2191+2195/2196/2197]		1
2	-	Padded Armrest [2198]		1
3	-	Inrigged Armrest [2183+2195/2196/2197/2199]		0
4	-	Flip-Up Armrest [2193+2195/2196/2197/2199]		0
5	-	Following Armrest [2192/2194+2195/2196/2197/2199]		0
6	-	Hip Support [2184/2185/2186/2187/2188/2189]		0

T-Armrest


1572570.iso

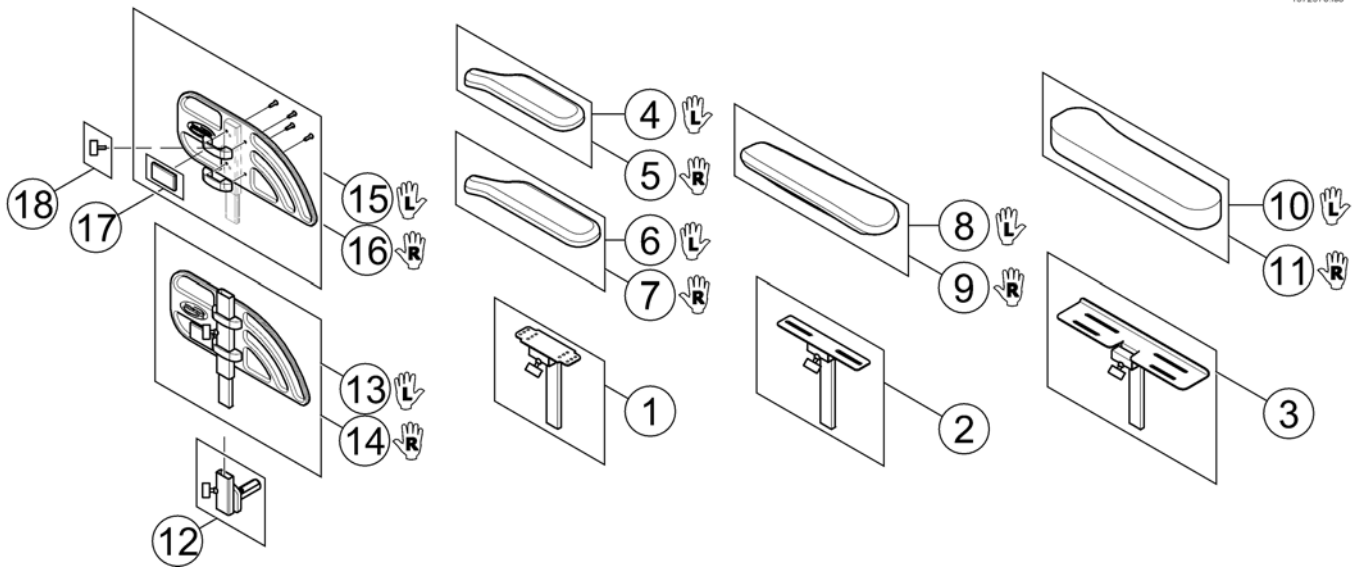


Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1567749	Siderest Bracket - LH/RH		1
13	SP1588374	Siderest LH		1
14	SP1588375	Siderest RH		1
15	SP1572940	Skirt Guard LH		1
16	SP1572948	Skirt Guard RH		1
17	SPP31298	Reflector, yellow		1
18	SPL10024	Wing Screw M6x16		1

Padded Armrest

1573214.00

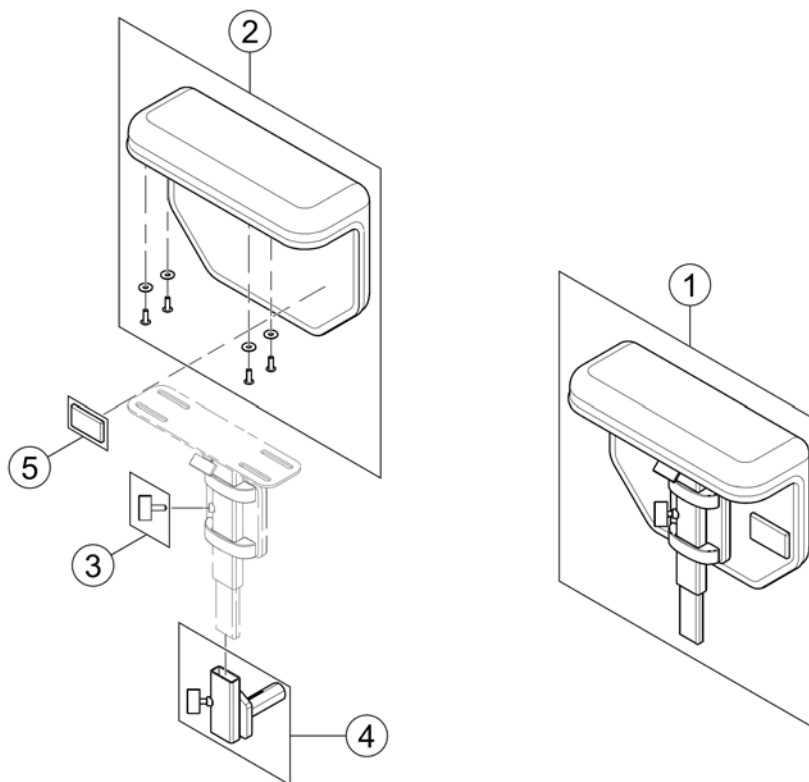


Pos.	Part number	Description	Valid from → until	Qty
1	SP1571987	Padded Armrest RH/LH, cpl.		1
2	SP1571962	Armpad, padded - RH/LH with integrated cloth guard		1
3	SPL10024	Wing Screw M6x16		1
4	SP1567749	Siderest Bracket - LH/RH		1
5	SPP31298	Reflector, yellow		1

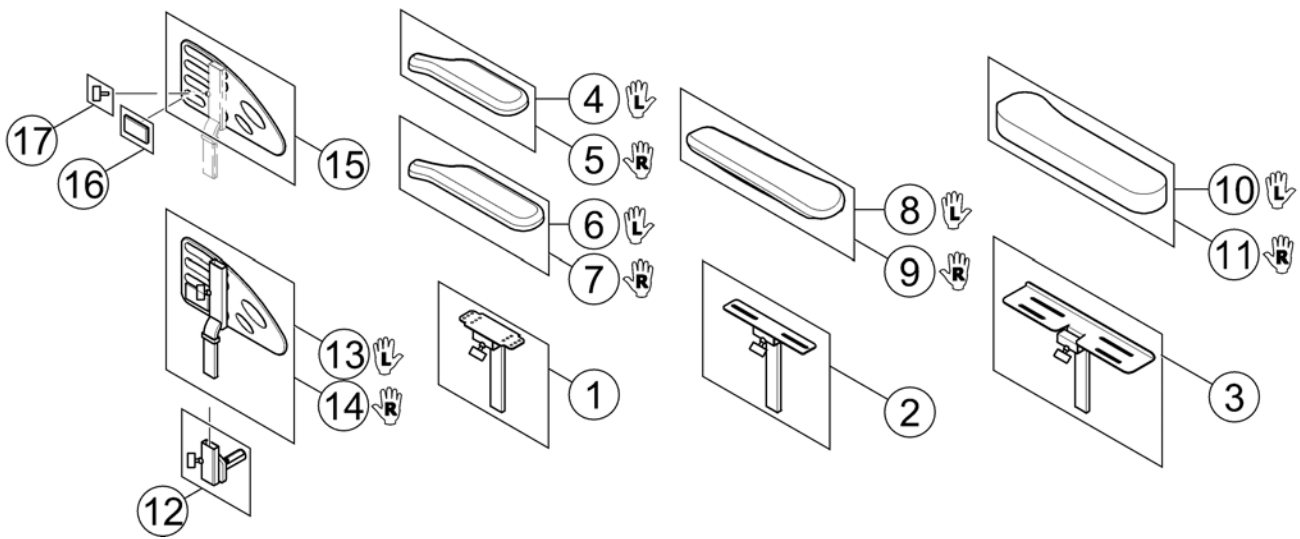

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1567749	Siderest Bracket - LH/RH		1
13	SP1585037	Siderest inrigged LH		1
14	SP1585038	Siderest inrigged RH		1
15	SP1578119	Skirt Guard LH/RH		1
16	SPP31298	Reflector, yellow		1
17	SPL10024	Wing Screw M6x16		1

Flip-Up Armrest

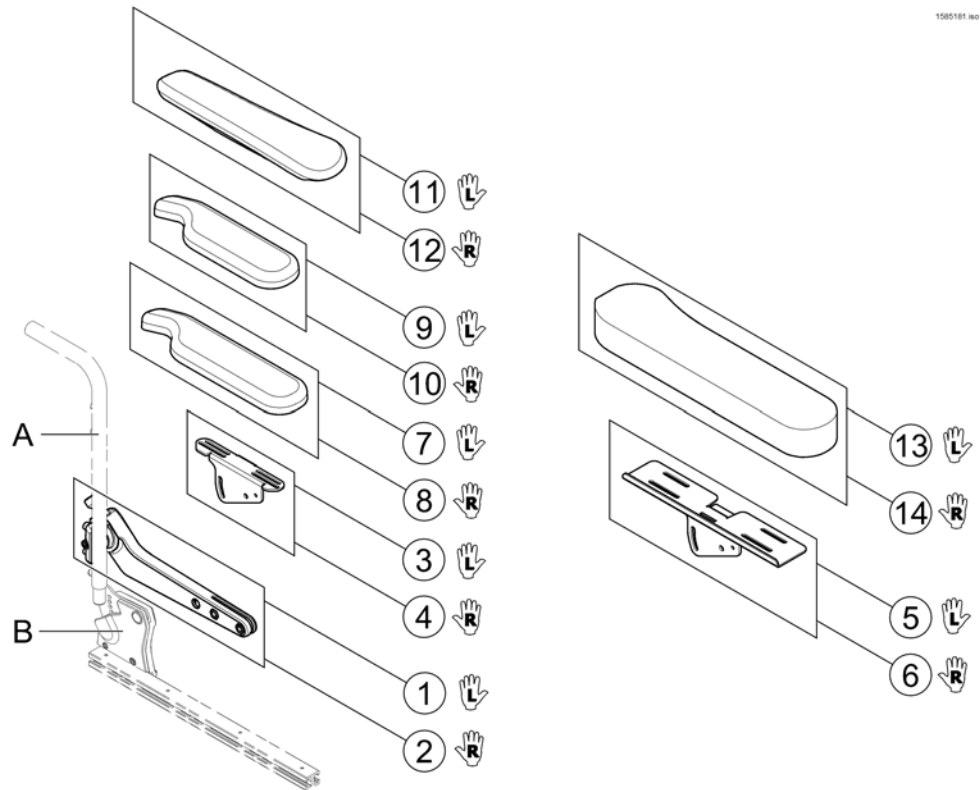


1580181 00

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1582668	Flip-Up Armrest, LH	2015-01-19 →	1
2	SP1582644	Flip-Up Armrest, RH	2015-01-19 →	1
3	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
4	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
5	SP1585414	Ampad Support (Soft), LH		1
6	SP1585415	Ampad Support (Soft), RH		1
7	SP1528846	Ampad, Extended (375mm), LH		1
8	SP1528845	Ampad, Extended (375mm), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1571589	Ampad, Ergonomic, LH		1
12	SP1571588	Ampad, Ergonomic, RH		1
13	SP1585068	Ampad, Soft, LH		1
14	SP1585069	Ampad, Soft, RH		1

Following Armrest


1585162 (No)

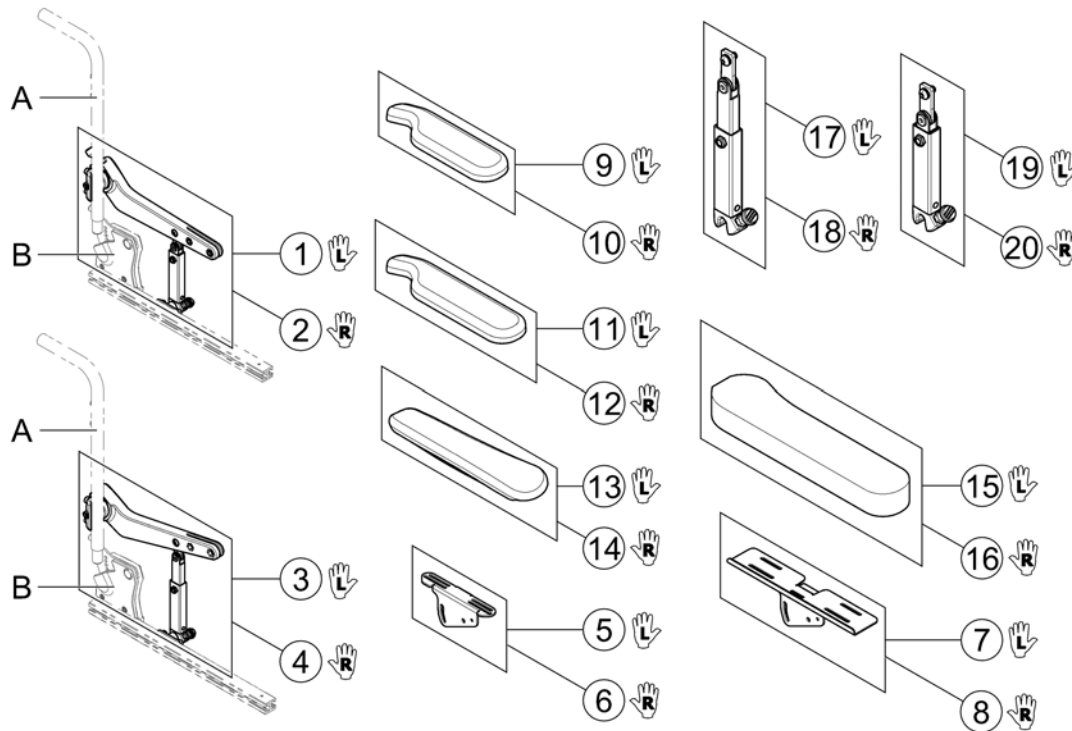

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1585263	Armrest LH cpl. - Short Support	2015-01-19 →	1
2	SP1585264	Armrest RH cpl. - Short Support	2015-01-19 →	1
3	SP1585267	Armrest LH cpl. - Long Support	2015-01-19 →	1
4	SP1585268	Armrest RH cpl. - Long Support	2015-01-19 →	1
5	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
6	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
7	SP1585414	Ampad Support (Soft), LH		1
8	SP1585415	Ampad Support (Soft), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1528846	Ampad, Extended (375mm), LH		1
12	SP1528845	Ampad, Extended (375mm), RH		1
13	SP1571589	Ampad, Ergonomic, LH		1
14	SP1571588	Ampad, Ergonomic, RH		1
15	SP1585068	Ampad, Soft, LH		1
16	SP1585069	Ampad, Soft, RH		1
17	SP1583156	Amrest Support, long, LH		1
18	SP1583155	Amrest Support, long, RH		1
19	SP1583154	Amrest Support, short, LH		1
20	SP1582639	Amrest Support, short, RH		1


Hip Support

158163 Rev

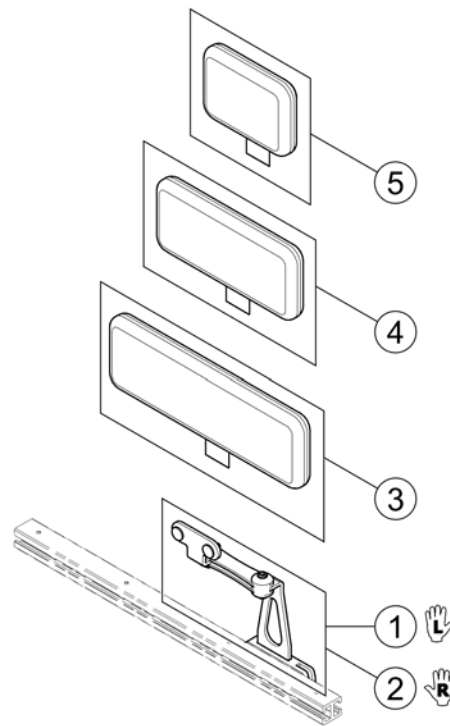

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584411	Hip Support, Quick Release bracket, LH		1
2	SP1584410	Hip Support, Quick Release bracket, RH		1
3	SP1584414	Hip Support Pad, large (330x100 mm)		1
4	SP1584413	Hip Support Pad, medium (230x100 mm)		1
5	SP1584412	Hip Support Pad, small (140x100 mm)		1


Front fork

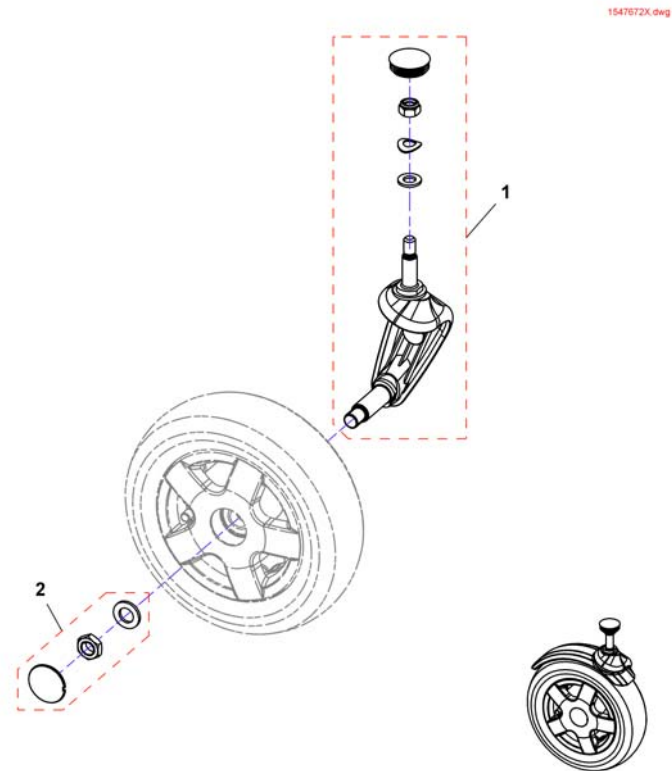

Pos.	Part number	Description	Valid from → until	Qty
1	SP1552920	Wheel Fork LH <i>we recommend to exchange the front forks by pairs for products built before 02.7.12, because of new design.</i>		1
1	SP1552926	Wheel Fork RH <i>we recommend to exchange the front forks by pairs for products built before 02.7.12, because of new design.</i>		1
2	SP1530554	Wheel Fixation Kit		1


Front fork, suspension

1551881X.iso

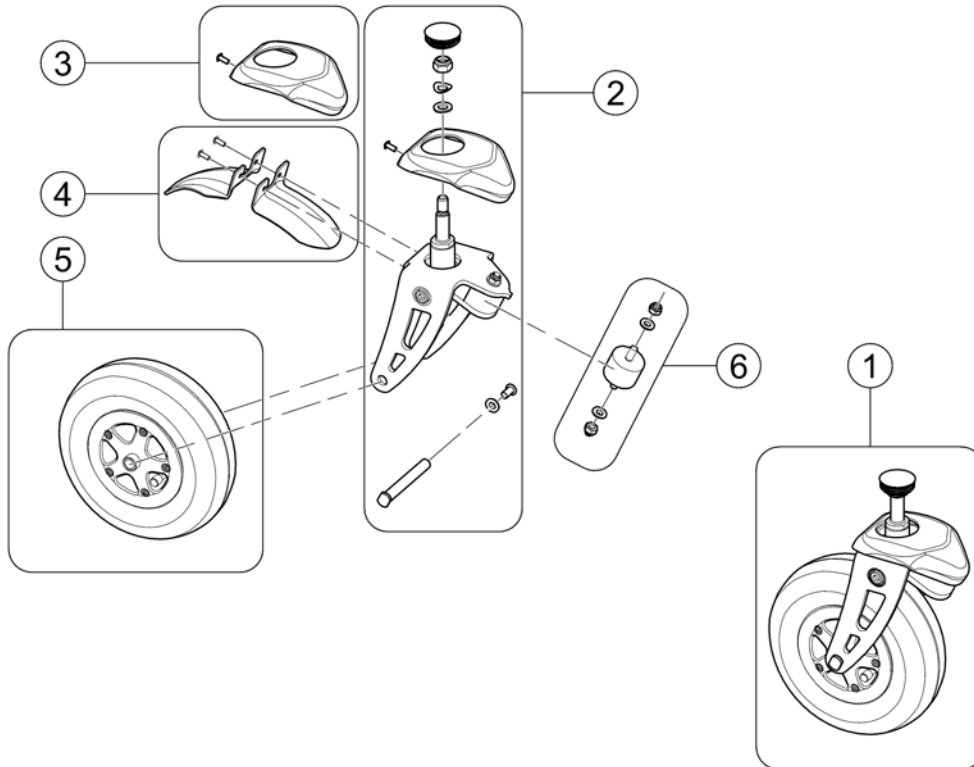


Pos.	Part number	Description	Valid from → until	Qty
1	SP1551412	Front fork, suspension, air, cpl., RH		1
1	SP1551644	Front fork, suspension, air, cpl., LH		1
1	SP1551677	Front fork, suspension, puncture proof, cpl., RH		1
1	SP1551678	Front fork, suspension, puncture proof, cpl., LH		1
1	SP1551883	Front fork, suspension, air, mud guard, cpl., RH <i>not shown</i>		1
1	SP1551881	Front fork, suspension, air, mud guard, cpl., LH <i>not shown</i>		1
1	SP1551882	Front fork, suspension, puncture proof, mud guard, cpl., RH <i>not shown</i>		1
1	SP1551880	Front fork, suspension, puncture proof, mud guard, cpl., LH <i>not shown</i>		1
2	SP1551413	Front fork, suspension		1
3	SP1551725	Upper cover		1
4	SP1551788	Mud guard, RH		1
4	SP1551822	Mud guard, LH		1
5	SP1478768	Wheel 230x60, puncture proof, grey, cpl. (C178-II NHS 2,80/2,50)		1
5	1483962	Wheel 230x60, pneumatic (C178-II NHS 2,80/2,50)		1
5	1513654	Tyre 230x60 (C178-II NHS 2,80/2,50)		1
5	SP1573940	Tyre Innova IA-2817-4 grey 2.80/2.50-4		1
5	1513655	Tube 230x60 (2,80/2,50-4 C178-II)		1
5	1513656	Tyre 230x60, puncture proof (C178-II Grey NHS 2,80/2,50-4)		1
5	SP1573941	Tyre Innova IA-2817-4 grey 2.80/2.50-4, puncture proof		1
6	SP1553708	Rubber-metal-bumper		1

Front wheels

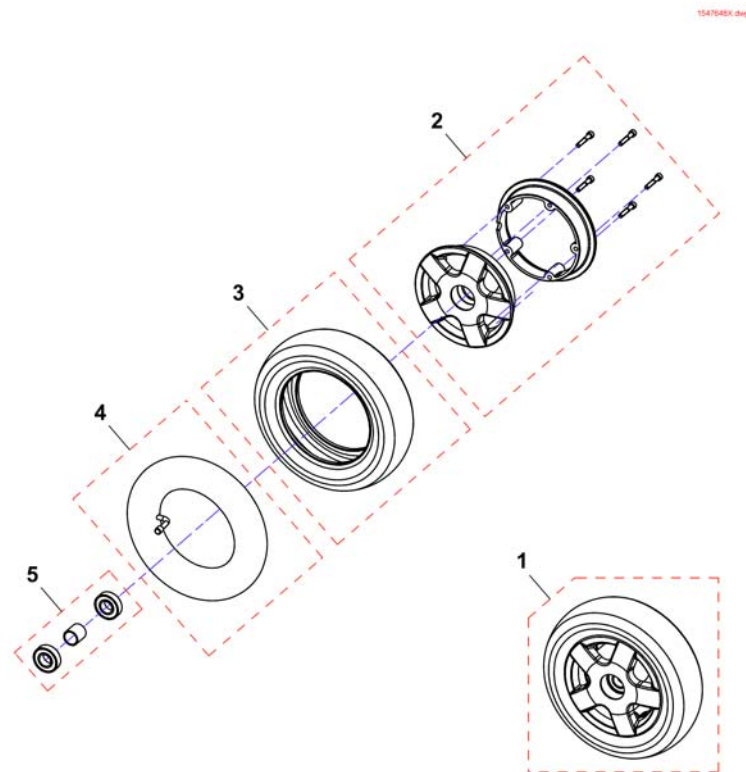

Pos.	Part number	Description	Valid from → until	Qty
1	SP1547837	Frontwheel, pneumatic, 10"		1
1	SP1547647	Frontwheel, puncture protect, 10"		1
1	SP1547648	Frontwheel, puncture proof, 10"		1
2	SP1423181	WHEEL RIM ASSEMBLY KIT 6", CPL. (for 10" rim)		1
3	SP1456309	Tyre 10" (for 6" rim)		1
4	1415175	TUBE 3.00 X 6 90 ° TR 87		1
5	SP1423182	WHEEL RIM BEARING SET 6", CPL.		1


Drive Wheels 14" (8x3.00)

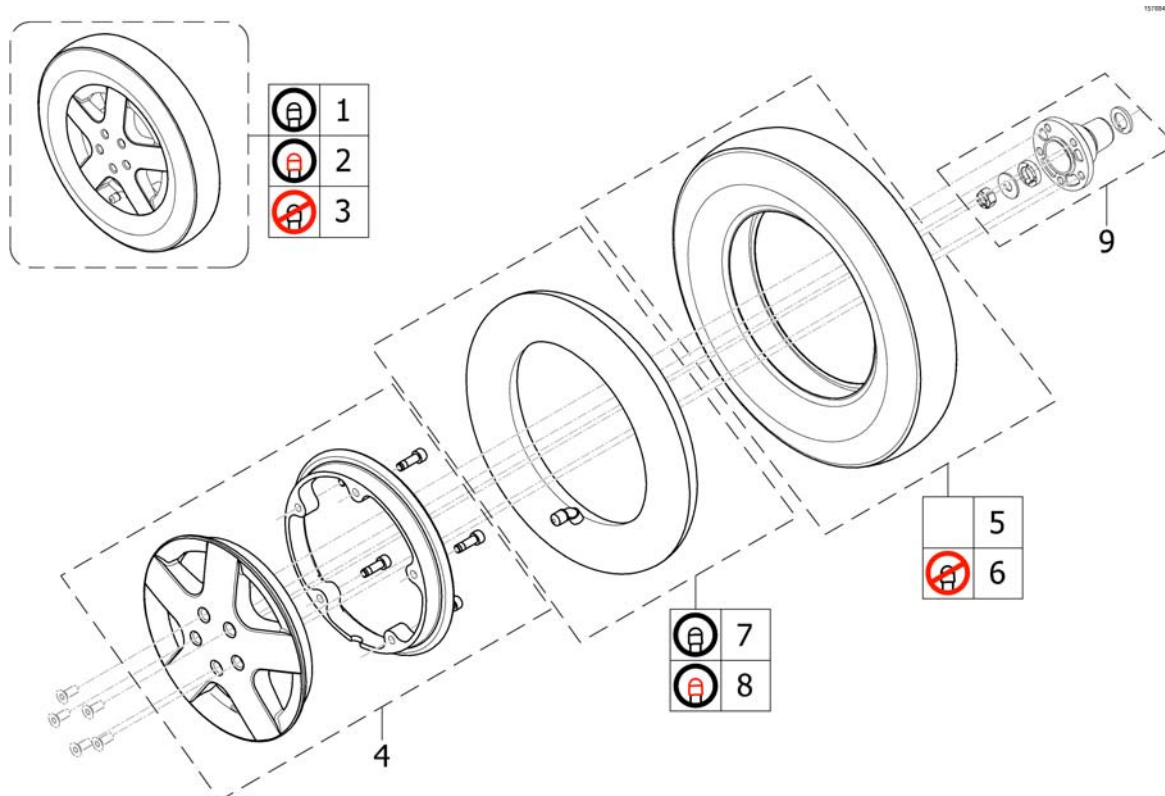

Pos.	Part number	Description	Valid from → until	Qty
1	SP1532414	Wheel 14 x 3.00" pneumatic, silver, complete		1
2	SP1532415	Wheel 14 x 3.00" puncture protected, silver, complete		1
3	SP1532416	Wheel 14 x 3.00" puncture proof, silver, complete		1
4	SP1541373	Rim Kit, complete		1
5	SP1522940	Tire 14 x 3.00"		1
6	SP1141342	Tire 14 x 3.00" puncture proof		1
7	SP1039651	Tube 14"		1
8	SP1541374	Tube 14" puncture protected		1
9	SP1570310	Wheel Hub Kit 5-hole, complete		1

Overview Propulsion SSD

1163096X.iso

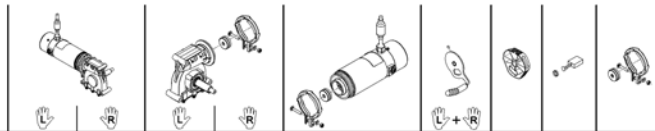

Bora/ SpectraXTR	6km/h	c	ACS/ Shark	1	2	33	34	37	---	51	45
			VR2	3	4	33	34	43	---	51	45
			R-Net	5	6	33	34	41	---	51	45
	10km/h	d	ACS/ Shark	7	8	31	32	36	---	51	45
			VR2	9	10	31	32	40	---	51	45
			R-Net	11	12	31	32	42	---	51	45
BoraHD/ Spectra XTR2HD	6km/h		ACS/ Shark	54	55	33	34	39	---	51	46
	10km/h		ACS/ Shark	56	57	31	32	38	---	51	46
	6km/h		R-Net	58	59	33	34	62	---	51	46
	10km/h		R-Net	60	61	31	32	63	---	51	46
FDX	6km/h	e		13	14	33	34	37	53	---	45
	6km/h	f	ACS/ Shark	54	55	33	34	39	53	---	46
	10km/h			15	16	31	32	38	53	---	46
Kite	6km/h		ACS/ Shark	17	18	33	34	37	---	51	45
	10km/h		ACS/ Shark	19	20	31	32	36	---	51	45
Kite HD	6km/h		ACS/ Shark	21	22	33	34	39	---	51	46
	10km/h		ACS/ Shark	23	24	31	32	38	---	51	46
Storm4/ Storm4 X-plore	6km/h		ACS/ Shark	47	48	34	33	39	---	51	46
	10km/h		ACS/ Shark	27	28	32	31	38	---	51	46
TDX-SP	6km/h	a		25	26	34	33	39	53	---	46
	8km/h	a	ACS/ Shark	29	30	34	33	38	53	---	46
	10km/h	b		49	50	32	31	38	53	---	46
TDX-SP Narrow	6km/h		ACS/ Shark	25	26	34	33	39	53	---	46
	10km/h		ACS/ Shark	49	50	32	31	38	53	---	46

a) used from 28.09.11 / b) used from 07.09.11 / c) used from 18.04.11 / d) used from 01.08.11 / e) used till 31.05.13 / f) used from 01.06.13 (S/N 13P06000078)

Pos.	Part number	Description	Valid from → until	Qty
-	N23004	Feather key DIN6885-A-6x6x40		1
1	SP1524195	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
2	SP1524194	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
3	SP1541408	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
4	SP1541410	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
5	SP1541409	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
6	SP1541411	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
7	SP1524197	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
8	SP1524196	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
9	SP1541404	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
10	SP1541406	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
11	SP1541405	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
12	SP1541407	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
13	SP1168983	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
14	SP1168984	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
15	SP1168989	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
16	SP1168988	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
17	SP1568147	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1

Overview Propulsion SSD


Pos.	Part number	Description	Valid from → until	Qty
18	SP1568148	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
19	SP1568149	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
20	SP1568150	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
21	SP1568151	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
22	SP1568152	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
23	SP1568153	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
24	SP1568208	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
25	SP1536022	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
26	SP1536021	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
27	SP1541366	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
28	SP1541365	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
29	SP1170307	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
30	SP1170306	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
31	SP1163413	Gearbox, LH, i = 18:1		1
32	SP1163412	Gearbox, RH, i = 18:1		1
33	SP1163415	Gearbox, LH, i = 23:1		1
34	SP1163414	Gearbox, RH, i = 23:1		1
35	SP1542953	Clamp band with clutch		1
36	SP1163099	Motor, 2-pole, 250W, fast, 600mm cable, ACS		1
37	SP1164046	Motor, 2-pole, 250W, slow, 600mm cable, ACS		1
38	SP1165863	Motor, 4-pole, 340W, fast, 760mm cable, ACS		1
39	SP1165865	Motor, 4-pole, 340W, slow, 760mm cable, ACS		1
40	SP1544840	Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
41	SP1544796	Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
42	SP1544841	Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
43	SP1544795	Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
45	SP1540358	Kit, brushes with caps, 2-pole (Set of 2)		1
46	SP1522967	Kit, brushes with caps, 4-pole (Set of 4)		1
47	SP1583821	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable <i>S/N 061122812868 =></i>		1
48	SP1583822	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable <i>S/N 061122812868 =></i>		1
49	SP1534060	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
50	SP1534059	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
51	SP1579595	Declutching knob SSD RHLH (1 pc) <i>Single part. Consider to buy another one for the other side if required. - Replaces SP1168668, SP1168669 & SP1169005</i>		1
53	SP1169004	Declutching lever, complete set (2 pcs.)		1
54	SP1552782	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1
55	SP1552783	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1

Overview Propulsion SSD

Pos.	Part number	Description	Valid from → until	Qty
56	SP1552872	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
57	SP1552873	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
58	SP1552832	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
59	SP1552833	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
60	SP1552836	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
61	SP1552837	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
62	SP1562498	Motor, 4-Pol, 340W, slow, 760mm cabel, R-Net		1
63	SP1562499	Motor, 4-Pol, 340W, fast, 760mm cabel, R-Net		1


Wheel Lock/Parkingbrake

1548717X.dwg

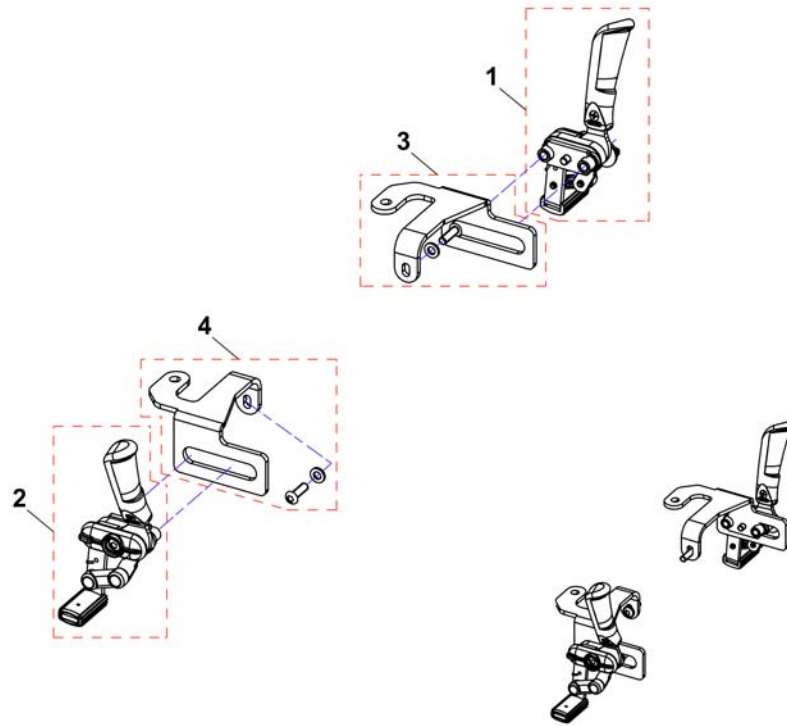

Pos.	Part number	Description	Valid from → until	Qty
1	SP1527950	Brake, LH		1
2	SP1527949	Brake, RH		1
3	SP1550695	Brakeholder, LH		1
4	SP1550696	Brakeholder, RH		1

1500784.indd

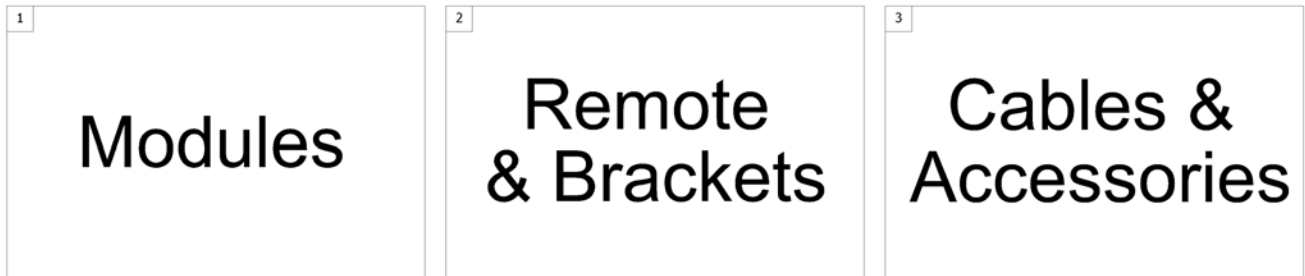

Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic Modules		1
2	-	ACS Remote & Brackets		1
3	-	ACS Cables & Accessories		1

ACS Electronic Modules

Pos.	Part number	Description	Valid from → until	Qty
-		ACS2 Powermodules		1
-		ACT for ACS 2		1
-		ACT-Brackets		1
-		G-Trac-sensor		1
-		Lightboard		1


ACS2 Powermodules

	ACS2 <small>PMACS2 100</small>			
	DX2-PMA70	DX2-PMA70L	DX2-PMA90L	DX2-PMA90LG
Bora/SpectraXTR	1	2	3	4
Dragon				
Dragon Vertic				
FDX	1	2	3	4
G40				
G50				
Kite	1	2	3	4
Mirage				
Mistral3				
Pronto M61				
Spectra Blitz	1			
Spectra Comfort	1	2		
Spectra Plus	1	2		
Spectra Seat Riser				
Storm ³				
Storm ³ GB				
Storm ³ Vertic				
Storm4			3	4
Storm4 MAX			3	4
Stream			3	4
TDX-SP				
Tiger	1	2		
Topan				
Tornado				
Typhoon				
Typhoon GB				
Typhoon II				
Zephyr				

Pos.	Part number	Description	Valid from → until	Qty
-	SP1555702	Gasket kit for ACS2, Powermodule and ACT		1
1	SP1528673	Powermodule, ACS2, 70A, DX2-PMA70 <i>Lock washer between module and bracket!</i>		1
2	SP1523433	Powermodule, ACS2, 70A, with light, DX2-PMA70L <i>Lock washer between module and bracket!</i>		1
3	SP1524244	Powermodule, ACS2, 90A, with light, DX2-PMA90L <i>Lock washer between module and bracket!</i>		1
4	SP1537975	Powermodule, ACS2, 90A, for light and G-Trac, DX2-PMA90LG <i>Lock washer between module and bracket!</i>		1


ACT for ACS 2

1528542X.iso

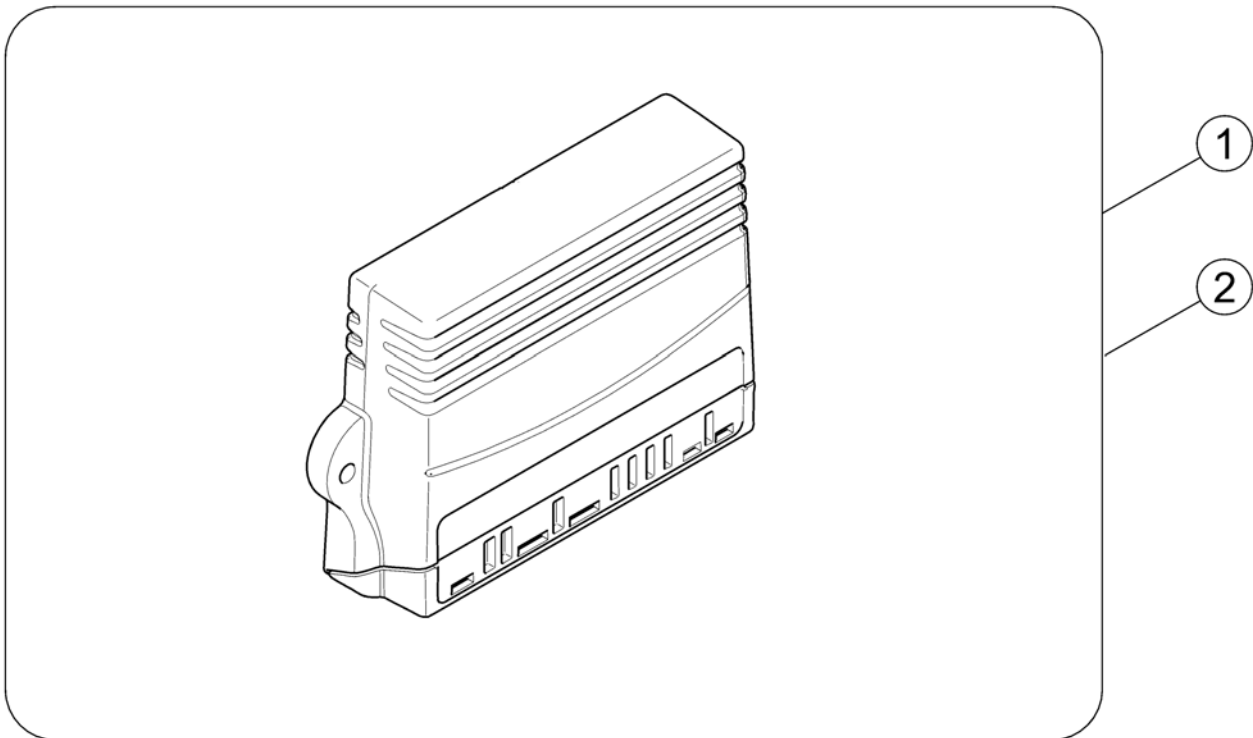

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528542	ACT2 for ACS2, cpl.		1
2	SP1529382	ACT4 for ACS2, cpl.		1


ACT-Brackets

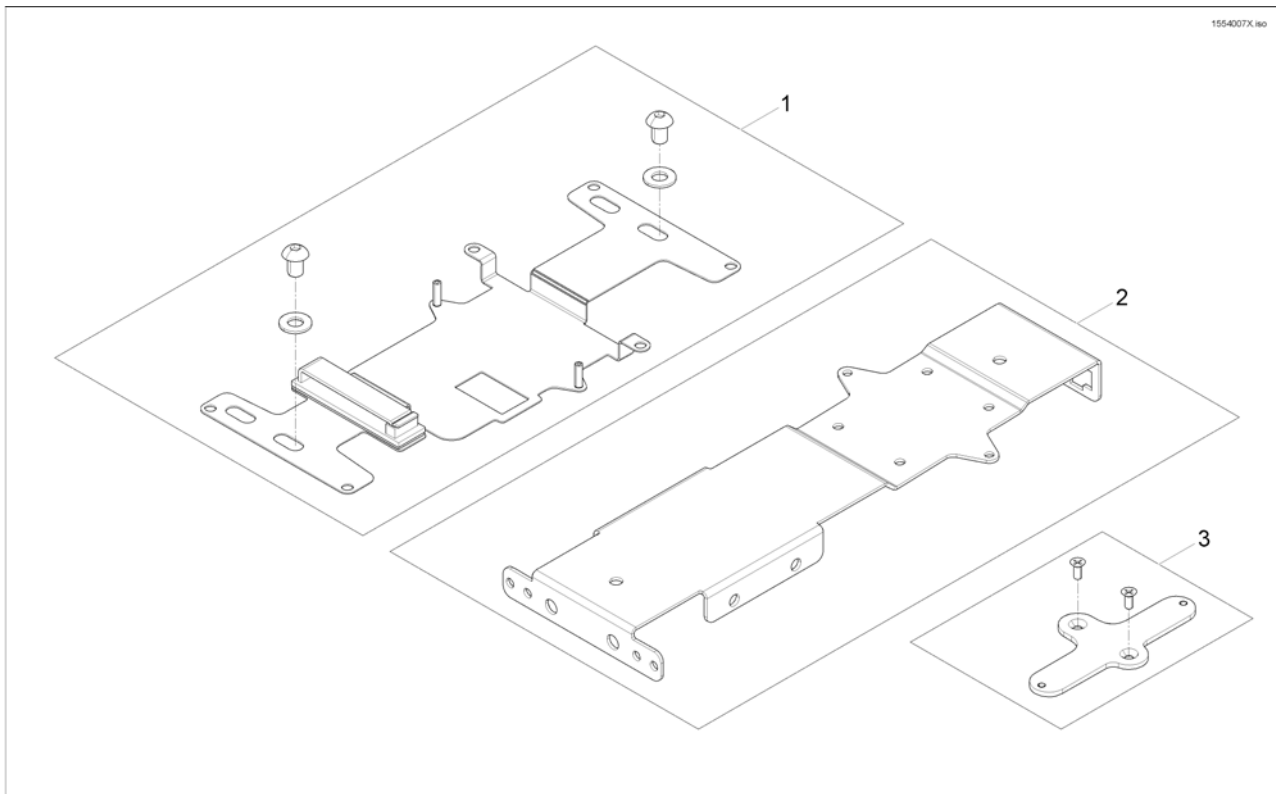


Pos.	Part number	Description	Valid from → until	Qty
1	SP1569960	ACT bracket		1
2	SP1564650	Multi support bracket		1
3	SP1554007	ACT cross bracket		1

G-Trac-sensor

1537848X.dwg

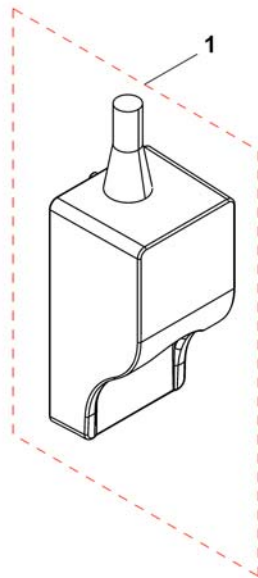


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541535	G-Trac		1

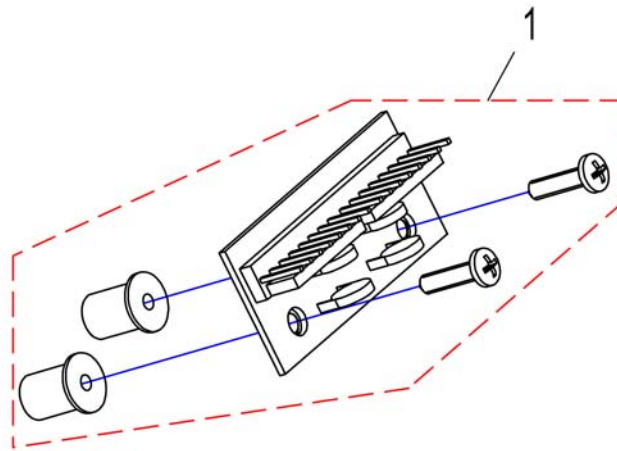


Pos.	Part number	Description	Valid from → until	Qty
1	SP1528550	Lightboard for all elektronik systems		1

ACS Remote & Brackets

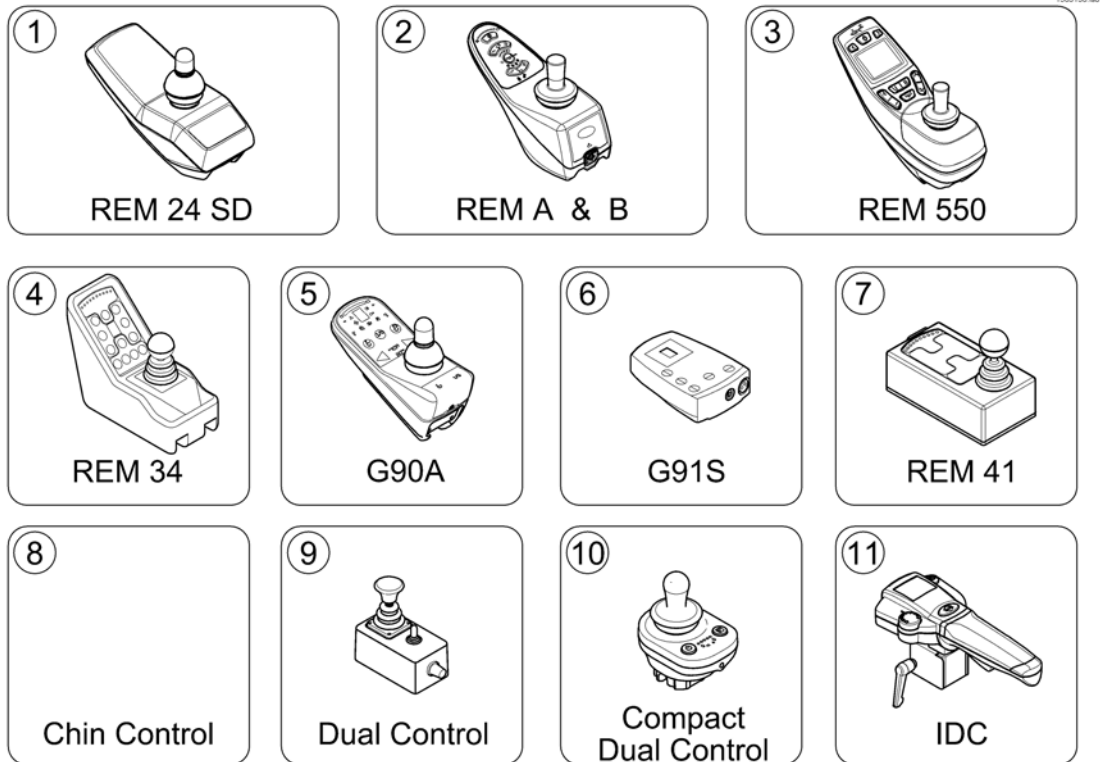


Pos.	Part number	Description	Valid from → until	Qty
1	-	REM 24 SD with brackets <i>[1601/1602]</i>		1
2	-	REM A & B with brackets <i>[1638/1639/1643/1644]</i>		1
3	-	REM 550 with brackets <i>[1617/1618]</i>		1
4	-	REM 34 with brackets <i>[1611]</i>		1
5	-	G90A with brackets <i>[1614/1616]</i>		1
6	-	G91S with brackets <i>[1615]</i>		1
7	-	Centre control REM 41 <i>[1607/1608]</i>		1
8	-	Small Chin Control with brackets <i>[1609]</i>		1
9	-	Dual Control with brackets <i>[1291/1292]</i>	→ 2015-06-30	1
10	-	Compact Dual Control with brackets <i>[1720]</i>	2015-07-01 →	1
11	-	Intuitive Dual Control with brackets <i>[1645]</i>		1

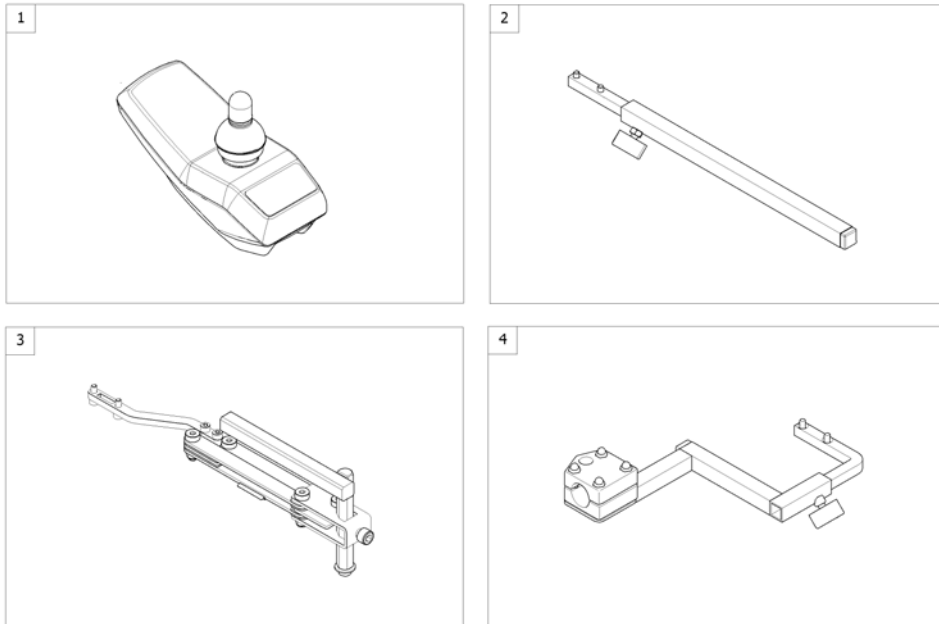


Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24SD [1601/1602]		0
2	-	Remote Bracket Standard [1601+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1601+1707/1708]		1
4	-	Remote Bracket Attendant [1602+1207/1208]		1

Remote REM 24SD

142802 8/0

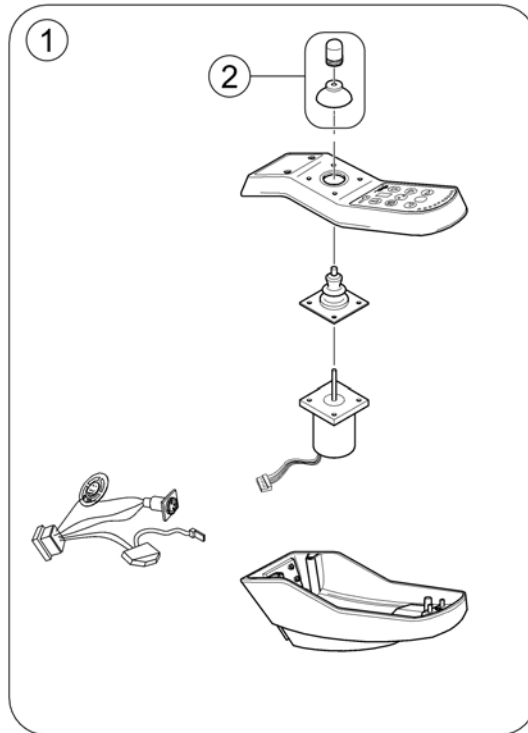


Pos.	Part number	Description	Valid from → until	Qty
1	1436541	Remote REM24SD		1
2	SPZ183229	Remote knob cpl.		1

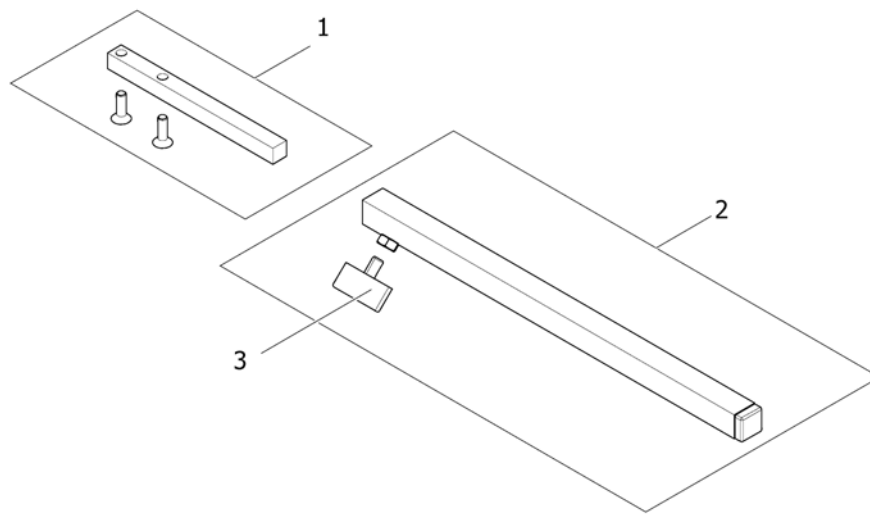


Pos.	Part number	Description	Valid from → until	Qty
1	SP1436542	Support, Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing Screw M6X10		1

Remote Bracket Swing away & height adjust.

1559852.iso

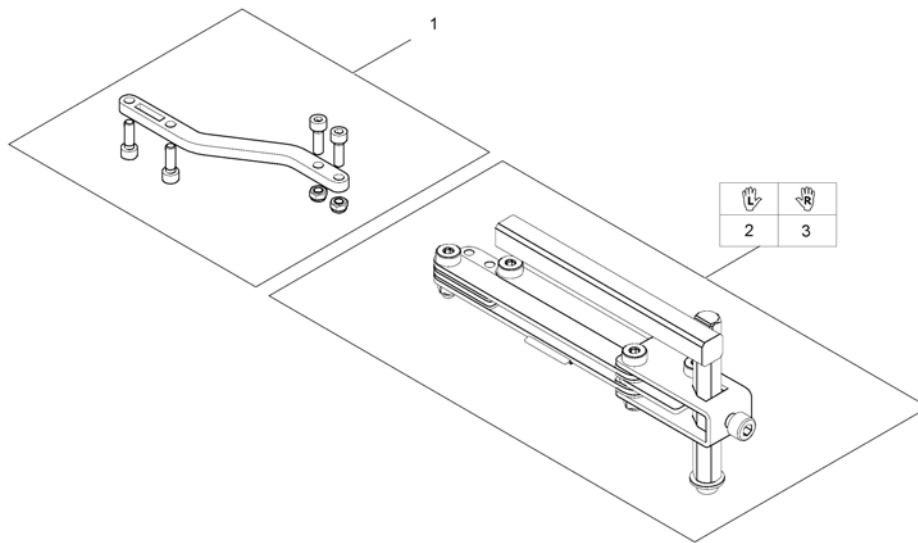


Pos.	Part number	Description	Valid from → until	Qty
1	SP1498222	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

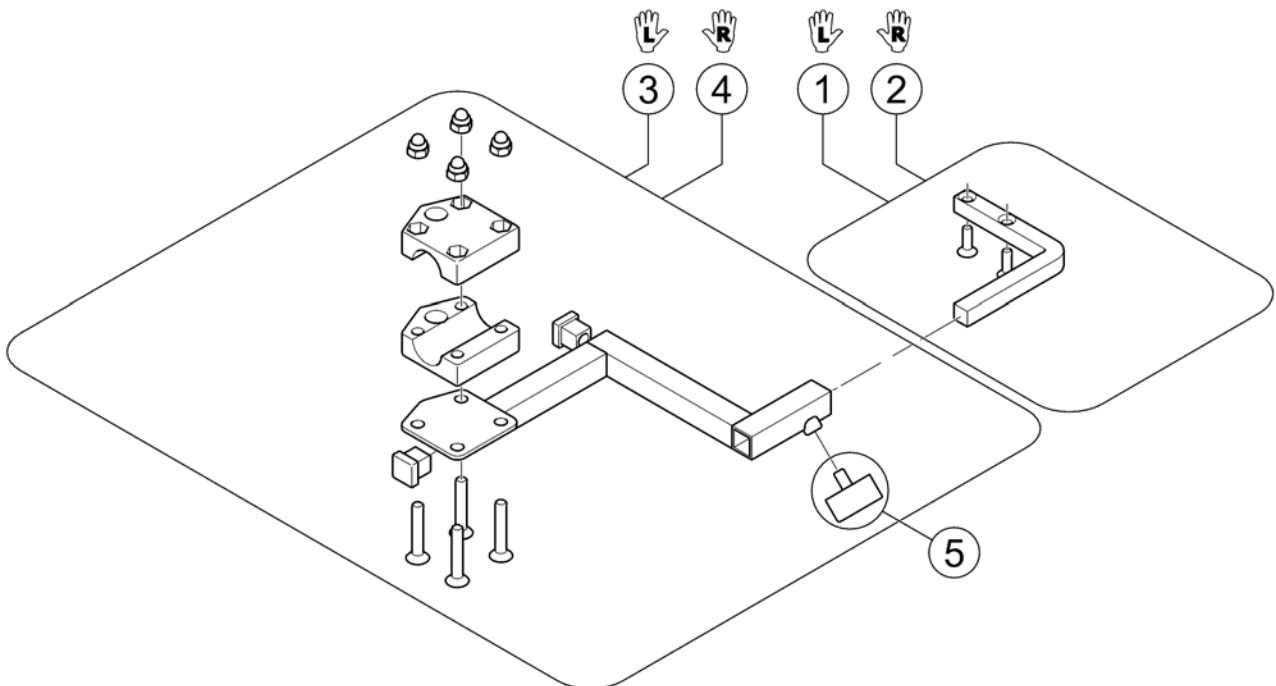

Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing Scew M6X10		1

REM A & B with brackets


1562687.iss

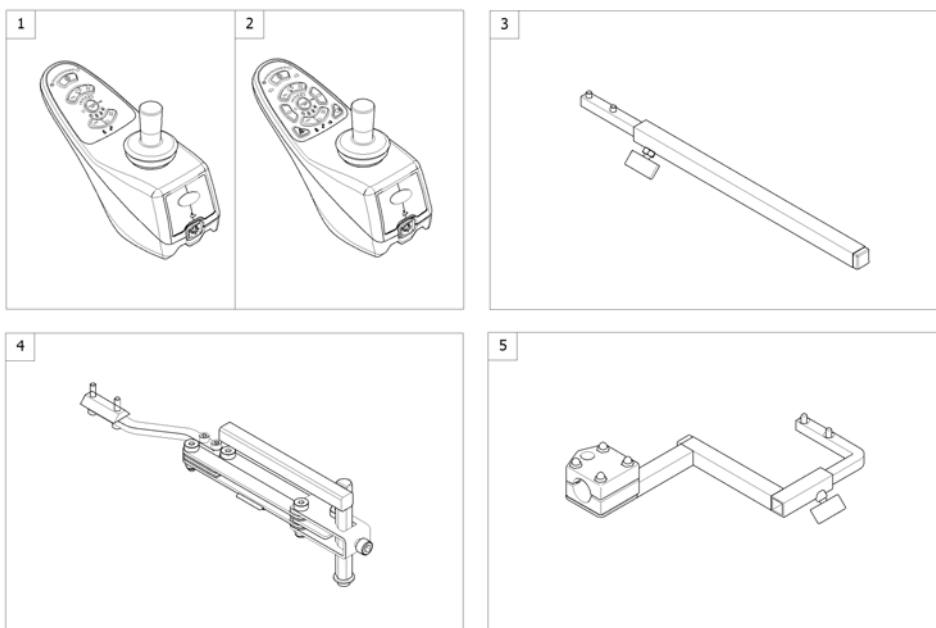

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523440	Remote "REM A" <i>[1638/1643]</i>		1
2	SP1523445	Remote "REM B" <i>[1639/1644]</i>		1
3	-	Remote Bracket Standard <i>[1638/1639+1207/1208]</i>		1
4	-	Remote Bracket Swing away & height adjust. <i>[1638/1639+1707/1708]</i>		1
5	-	Remote Bracket Attendant <i>[1643/1644+1207/1208]</i>		1

Remote Bracket Swing away & height adjust.


1559857.iso

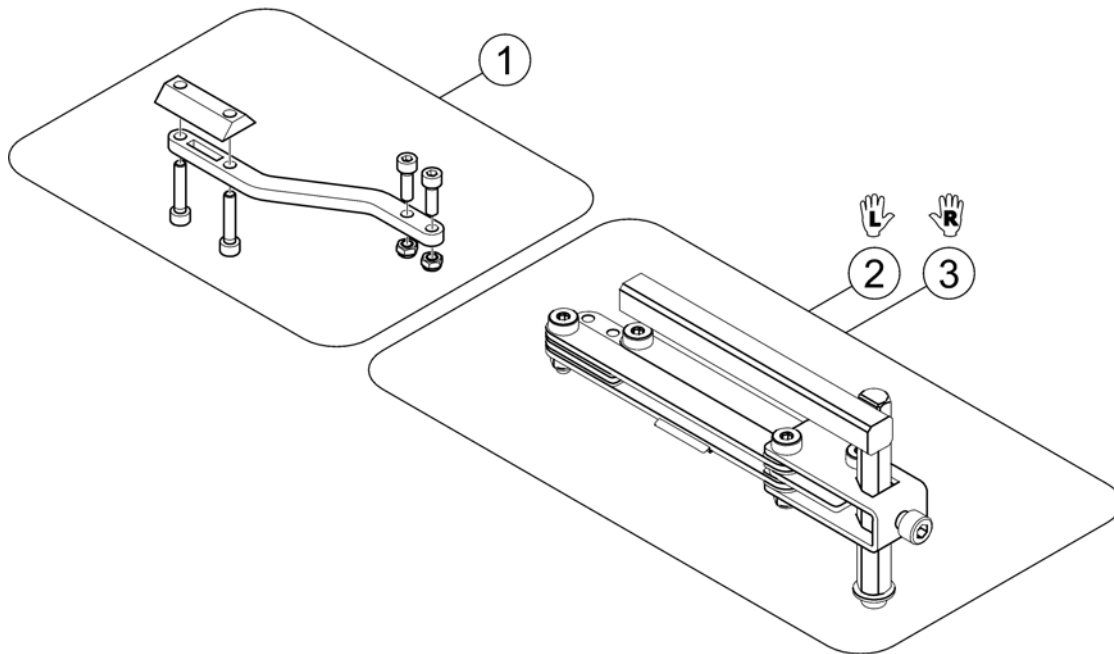

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536180	Support for Remote Bracket, height adjustable & swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1



REM 550 with brackets

1562658.iss

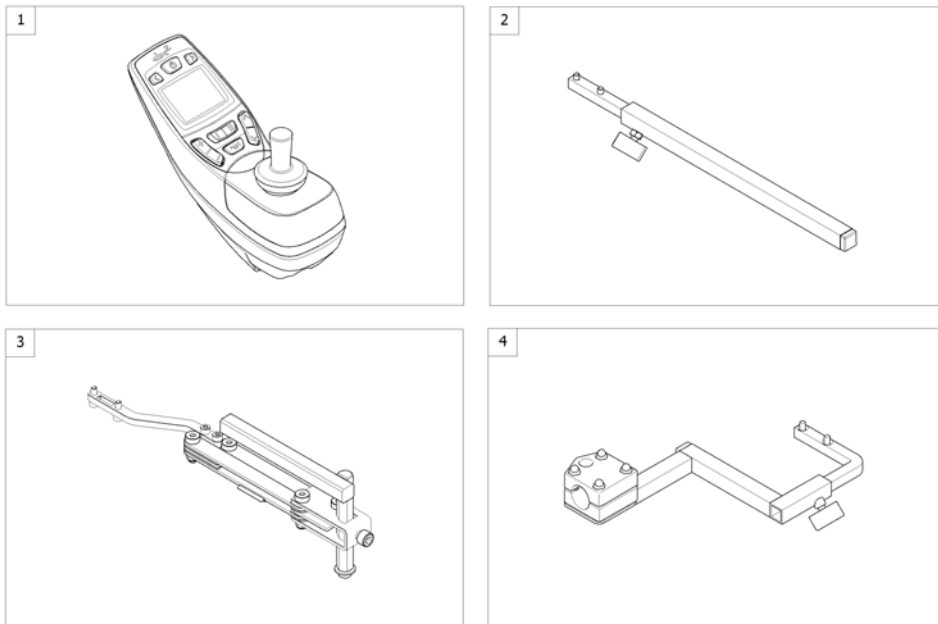

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523444	Remote REM550 <i>[1617/1618]</i>		1
2	-	Remote Bracket Standard <i>[1617+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1617+1207/1208]</i>		1
4	-	Remote Bracket Attendant <i>[1618+1207/1208]</i>		1


1560846 iso

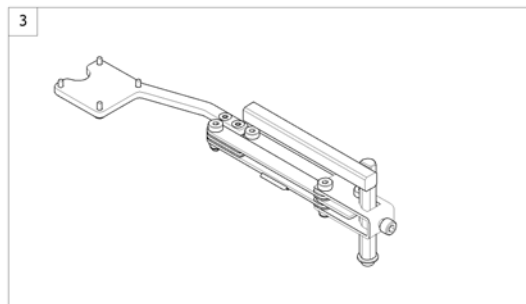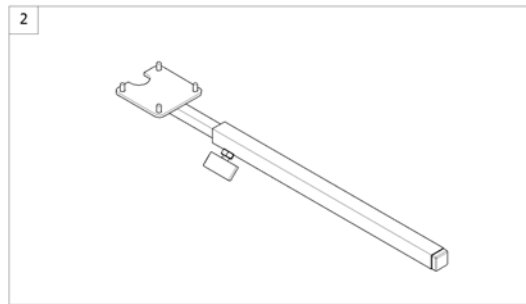

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440677	Remote REM 34 [1611]		1
2	-	Remote Bracket Standard [1611+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1611+1707/1708]		1

Remote Bracket Standard


1560844.iso

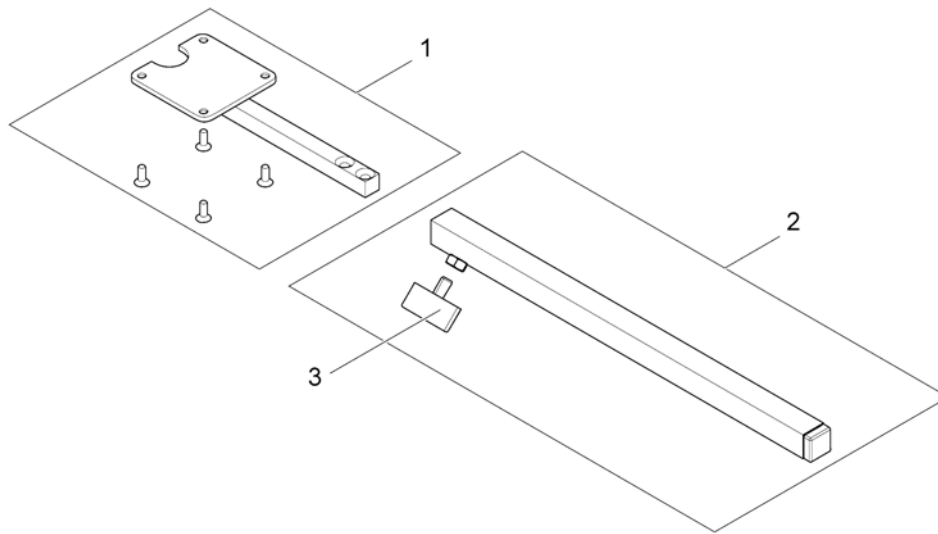

Pos.	Part number	Description	Valid from → until	Qty
1	SP1434921	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1560845.iso

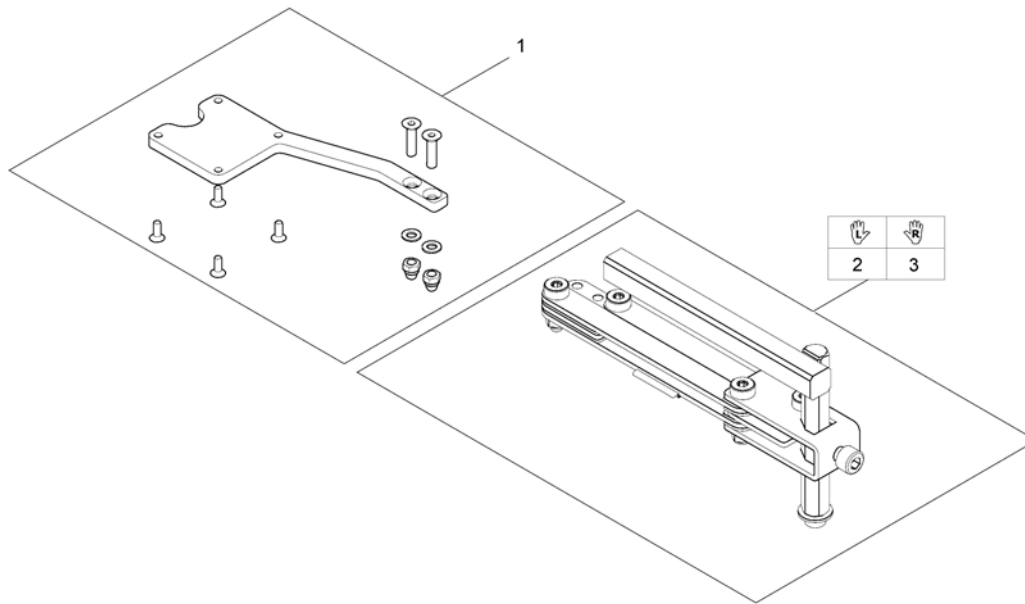


Pos.	Part number	Description	Valid from → until	Qty
1	SP1527125	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

G90A with brackets

1562659.iss

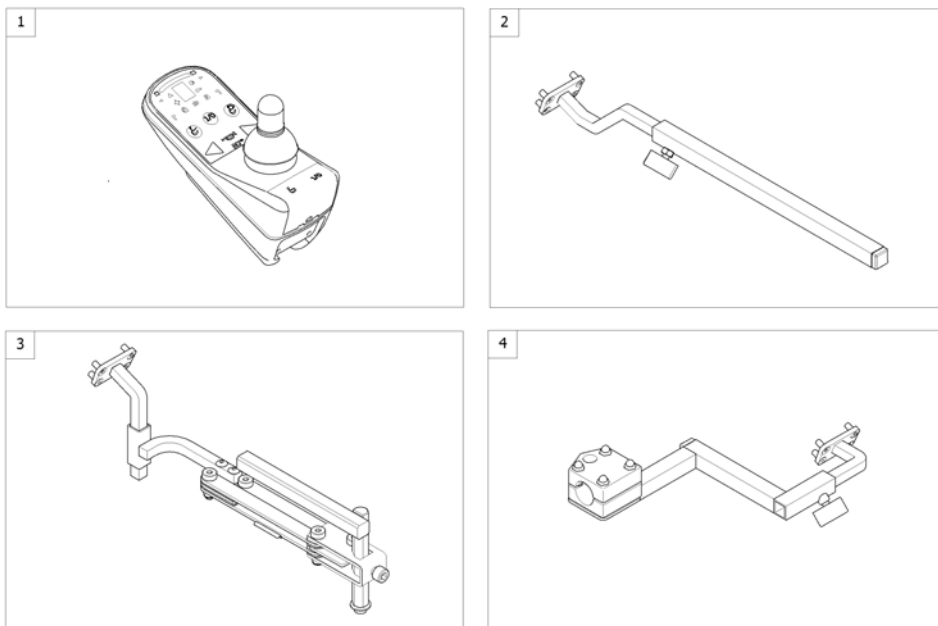


Pos.	Part number	Description	Valid from → until	Qty
1	SP1504720	Remote G90A <i>[1614/1616]</i>		1
2	-	Remote Bracket Standard <i>[1614+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1614+1707/1708]</i>		1
4	-	Remote Bracket Attendant <i>[1616+1207/1208]</i>		1

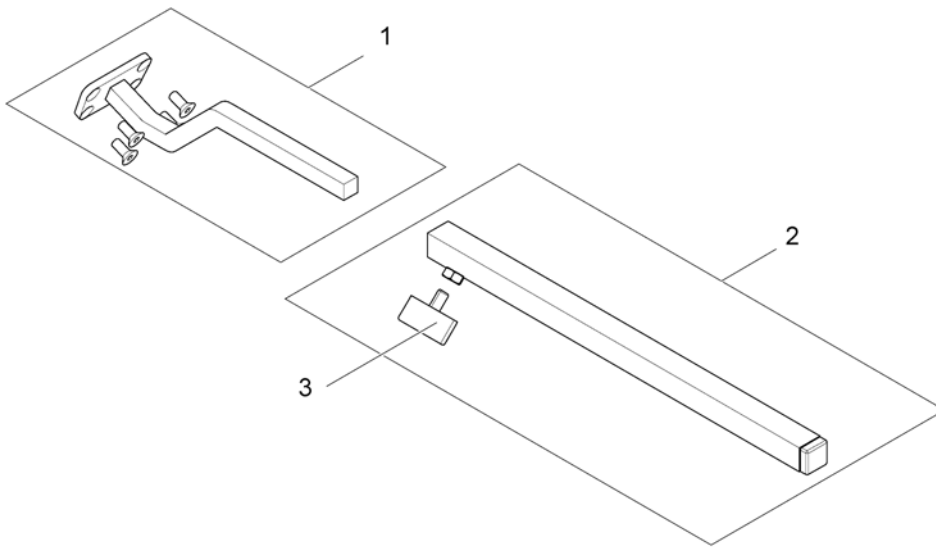


Pos.	Part number	Description	Valid from → until	Qty
1	SP1504822	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.

1560310.iso

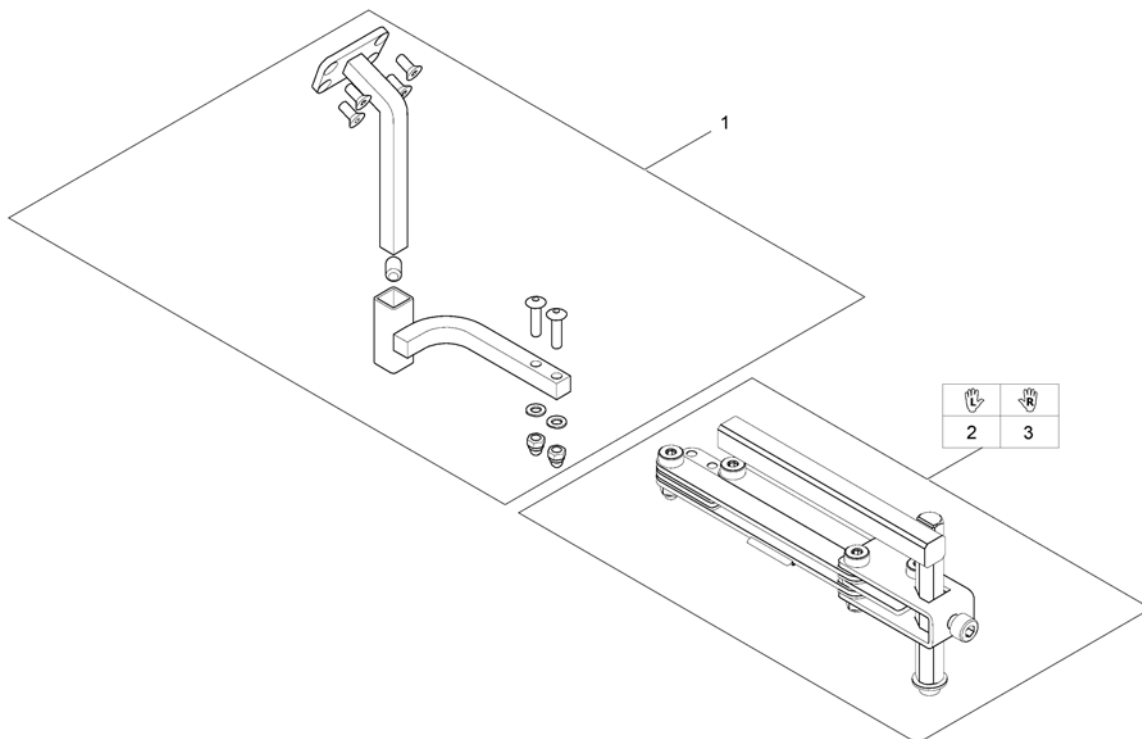



Pos.	Part number	Description	Valid from → until	Qty
1	SP1517352	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

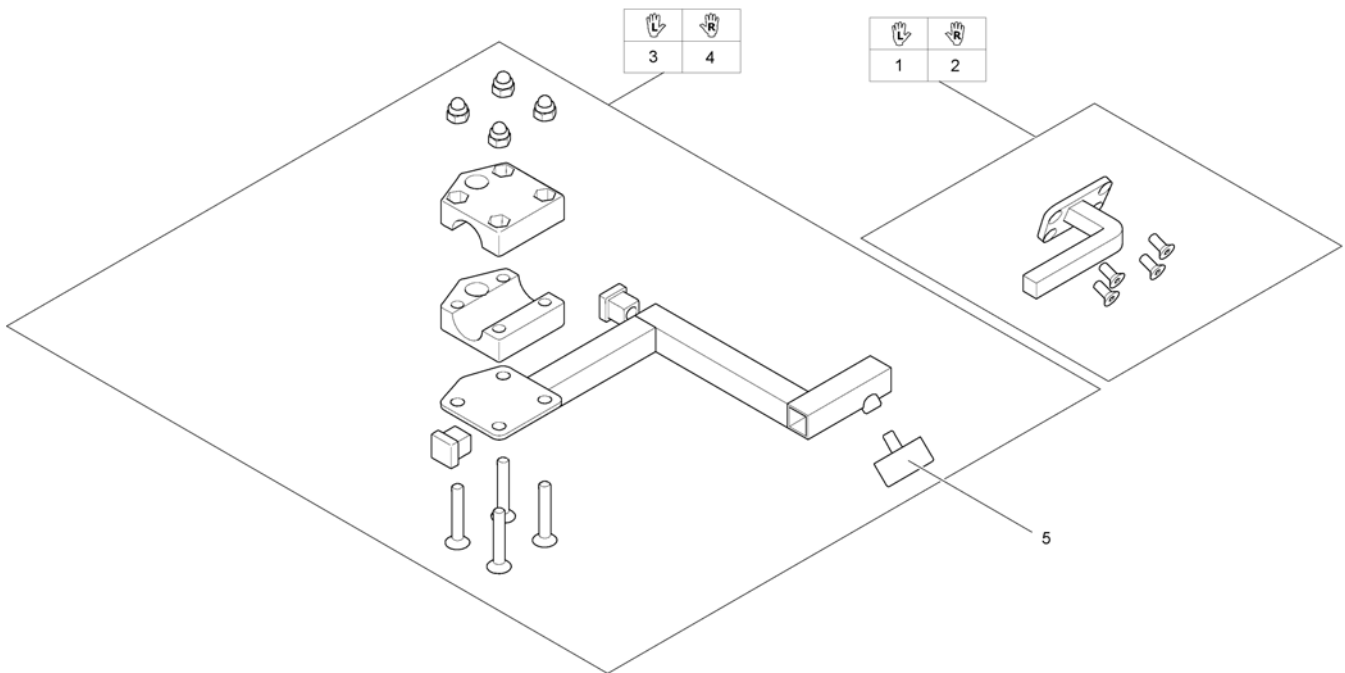

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1


G91S with brackets

1560850 iso

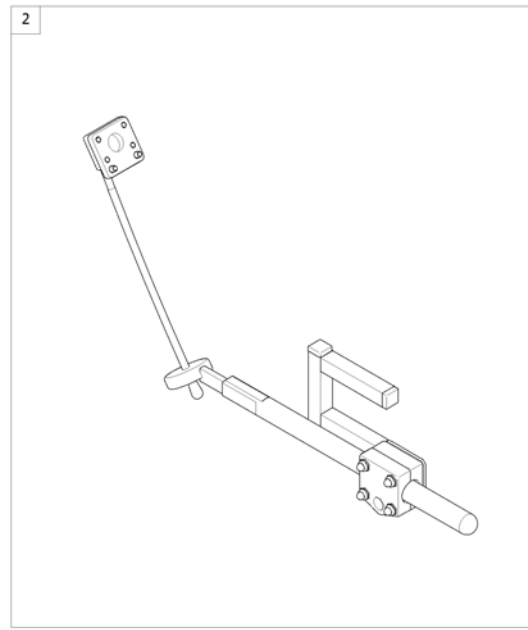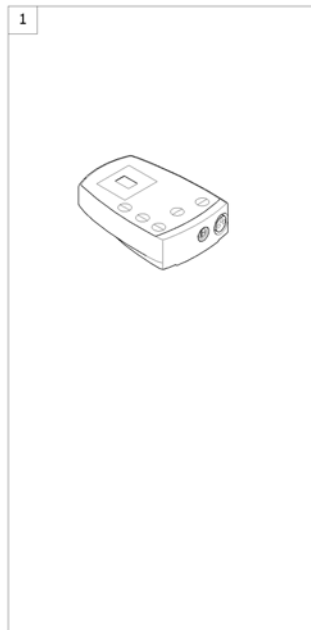

Pos.	Part number	Description	Valid from → until	Qty
1	SP1505187	Remote Control G91S <i>[1615]</i>		1
2	-	Remote Bracket Standard <i>[1615+1207/1208]</i>		1


1560654.iso

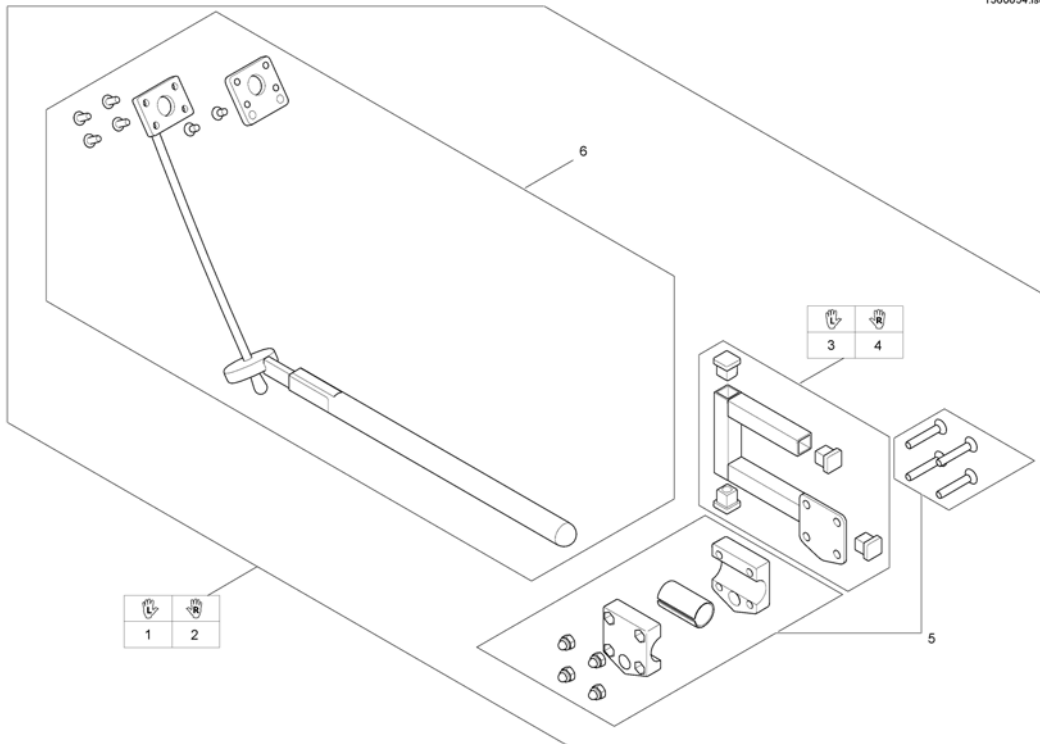

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509718	Remote Bracket G91/G91S LH, cpl.		1
2	SP1509690	Remote Bracket G91/G91S RH, cpl.		1
3	SP1509716	Adapter G91 LH		1
4	SP1509711	Adapter G91 RH		1
5	SP1527503	Gripper clamp cpl.		1
6	SP1527504	Swan neck remote bracket with adapter plate		1

Centre control REM 41

F383057.dwg

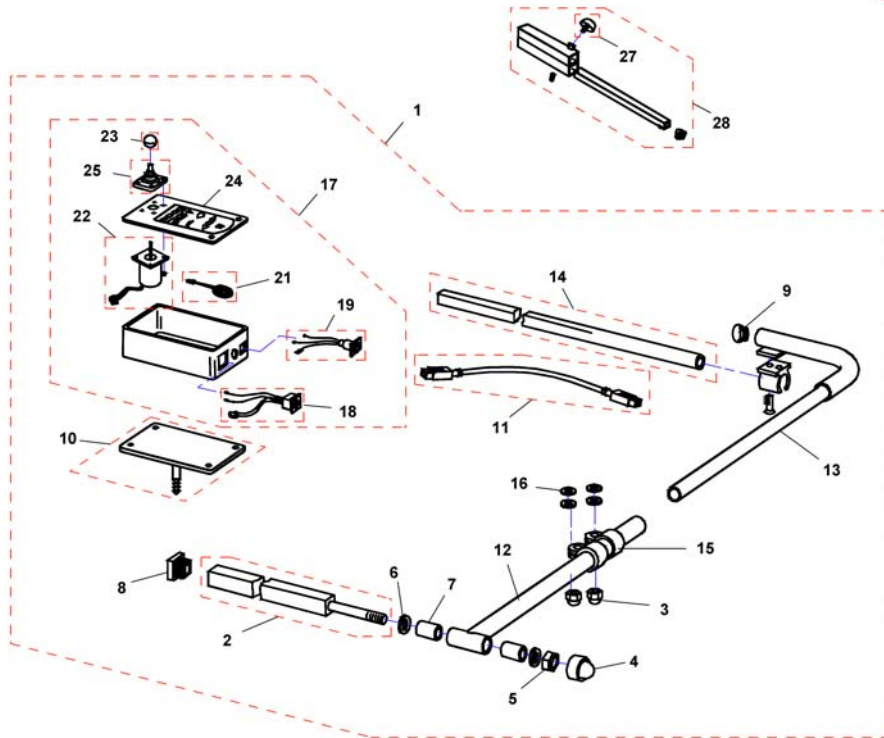


Pos.	Part number	Description	Valid from → until	Qty
-	Q99081	User Manual Table- & Middle Control REM41-S German		1
1	SP1530588	CENTRAL STEERING CPL.; SWING-AWAY RH, REM41-D		1
1	SP1530589	CENTRAL STEERING CPL.; SWING-AWAY LH, REM41-D		1
2	Z177024.00	SUPPORT TUBE; CHROME PLATED		1
3	F60208	Cap nut DIN1587-M08 zc		1
4	L01037	Sks Nut Cap M12		1
5	F20106	Hexagon nut DIN985-M12 zc		1
6	G00104	Washer DIN125-Ø13 zc		1
7	Z17032A	Bush Rd. 19 X 3.5 X 25		1
8	L01055	Plug 15x15mm		1
9	L01008	Tube End Plug SI 22		1
10	Z183216.P40	SUPPORT BRACKET; RAL 9005 BLACK		1
11	1559862	Bus cable ACS, 1700mm		1
12	Z183089.00	SWING ARM; CHROME PLATED		1
13	Z183086.00	SUPPORT ARM RH; CHROME PLATED		1
13	Z183092.00	SUPPORT ARM LH; CHROME PLATED		1
14	H281005A.00	MIRROR SUPPORT RAIL; CHROME PLATED		1
15	5006720	Klemmschelle F. Dm 22 mm		1
16	G00106	Washer DIN125-A-Ø8,4 zc		1
17	SP1527930	DX ACS Tray Remote DX-REM41		1
18	P41108	ACS-BUS CABLE SOCKET		1
19	P41106	CHARGER SOCKET		1
21	P30493	HORN		1
22	GSM55120	ACS-remote WITH 180MM CABLE		1
23	L36999	KNOB FOR remote 32 MM, HOLE 5.8 MM		1
24	-	No longer available.		0
25	Z183001A	Rubber gaiter for ACS remote		1
26	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1
27	L31001	Wing ScREW M6X10		1
28	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1

Small Chin Control with brackets


Pos.	Part number	Description	Valid from → until	Qty
-		Small Chincontrol		1
-		Framework for chincontrol		1

Small Chincontrol

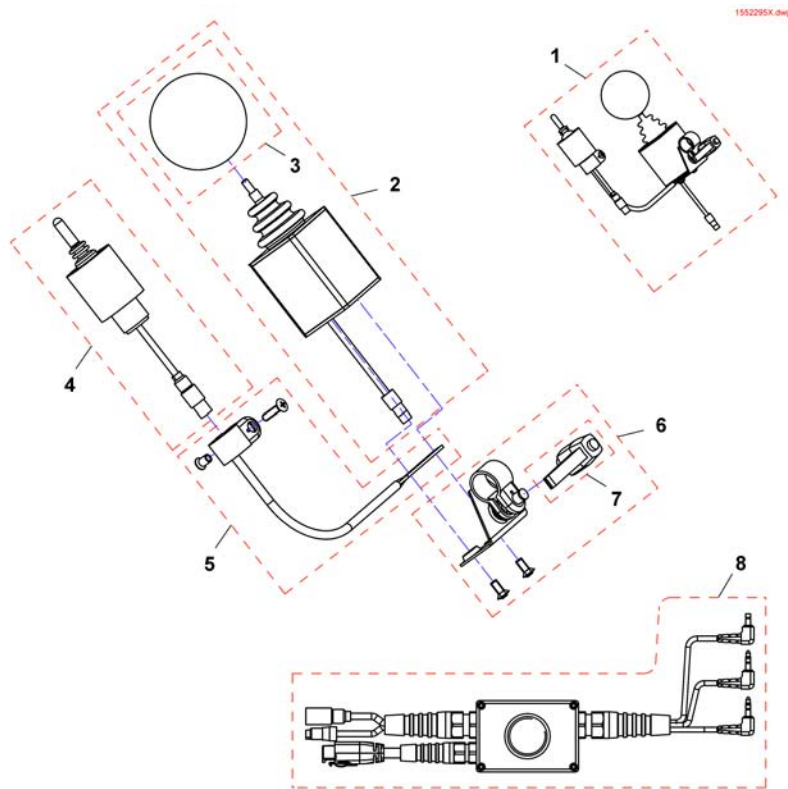

Pos.	Part number	Description	Valid from → until	Qty
1	1448705	Chin control REM24, cpl.		1
1	1421770	Chin control G80/G90, cpl.		1
1	SP1544255	Chin control G91, cpl.		1
2	Z183282	Joystick, kmpl.		1
3	Z183265	Foam control knob cpl.		1
4	1442943	Switchjoystick REM24, kmpl.		1
4	Z183283	Switchjoystick G80/G90, cpl.		1
4	SP1544250	Switchjoystick G91, kmpl.		1
5	SP1552294	Switchjoystick bracket		1
6	SP1552295	Joystick clamp		1
7	L32901	Clamping lever		1
8	SP1442944	Harness REM24		1
8	Z183275	Harness G80/G90		1
8	SP1541715	Harness G91		1




Framework for chincontrol

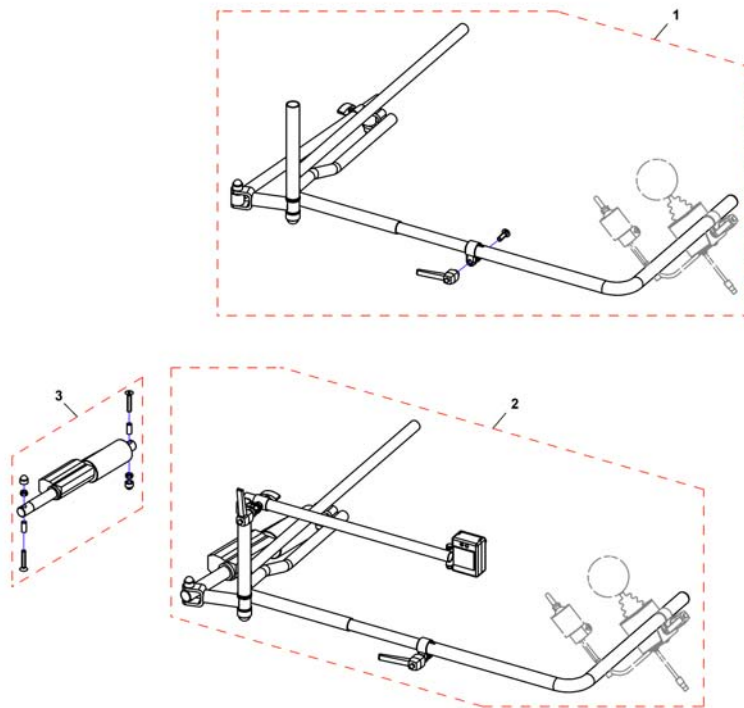

Pos.	Part number	Description	Valid from → until	Qty
-	Z167994	CHIN CONTROL SUPPORT . CPL. <i>Not shown / Nicht abgebildet / Non illustré</i>		1
1	SP1422943	Framework for chin control, manual, horizontal swing away, LH, cpl.		1
1	SP1422942	Framework for chin control, manual, horizontal swing away, RH, cpl.		1
2	SP1422945	Framework for chincontrol, powered, horizontal swing away, LH, cpl. <i>incl. Actuator</i>		1
2	SP1422944	Framework for chin control, powered, horizontal swing away, RH, cpl. <i>incl. Actuator</i>		1
-	SP1552335	Hardware, chin control, powered, horizontal swing away, LH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1552336	Hardware, chin control, powered, horizontal swing away, RH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1422946	Framework for chincontrol, powered, vertikal swing away <i>Not shown / Nicht abgebildet / Non illustré</i>		1
3	SP1552337	Actuator for horizontal swing away chin control, cpl.		1
3	SP1552418	Gestell für Kinnbedienung, elektrisch, vertikal abschwenkbar <i>Not shown / Nicht abgebildet / Non illustré</i>		1

Dual Control with brackets


1574292 iso

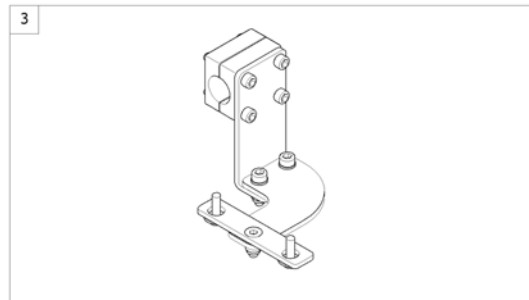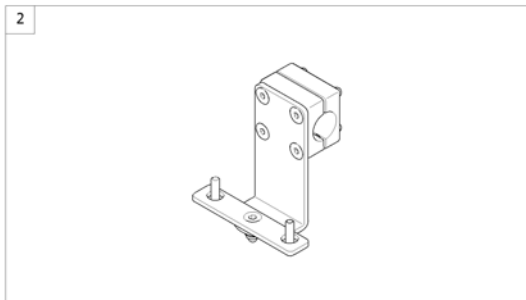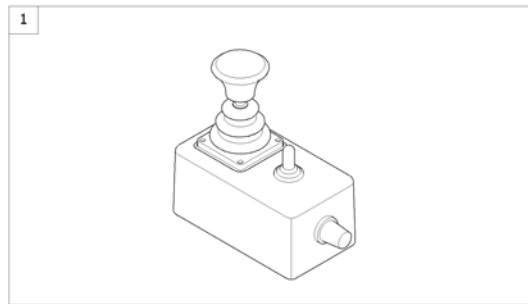

Pos.	Part number	Description	Valid from → until	Qty
1	6001425	ACS dual control <i>[1291/1292]</i>		1
2	-	Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat		1
3	-	Remote Bracket Dual Control for Modulite Seat		1

Remote Bracket Dual Control for Standard-/Flex2-/Max-Seat


1560885.iso

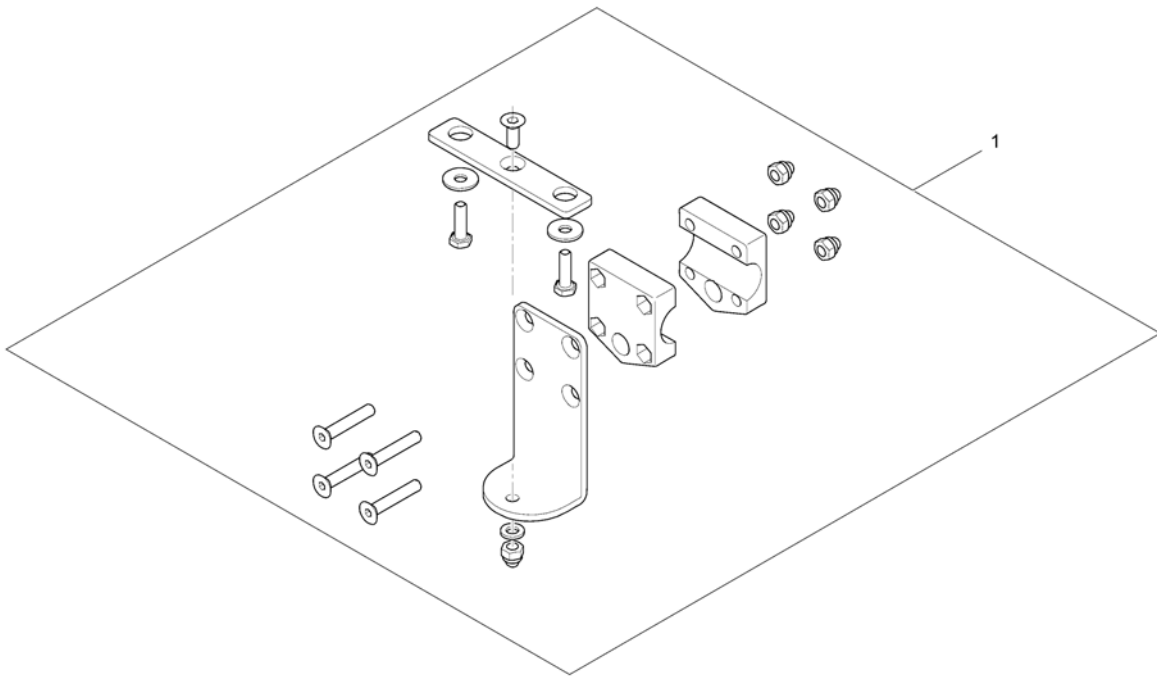

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422818	Remote Bracket Dual Control		1

Remote Bracket Dual Control for Modulite Seat


1574291 iso

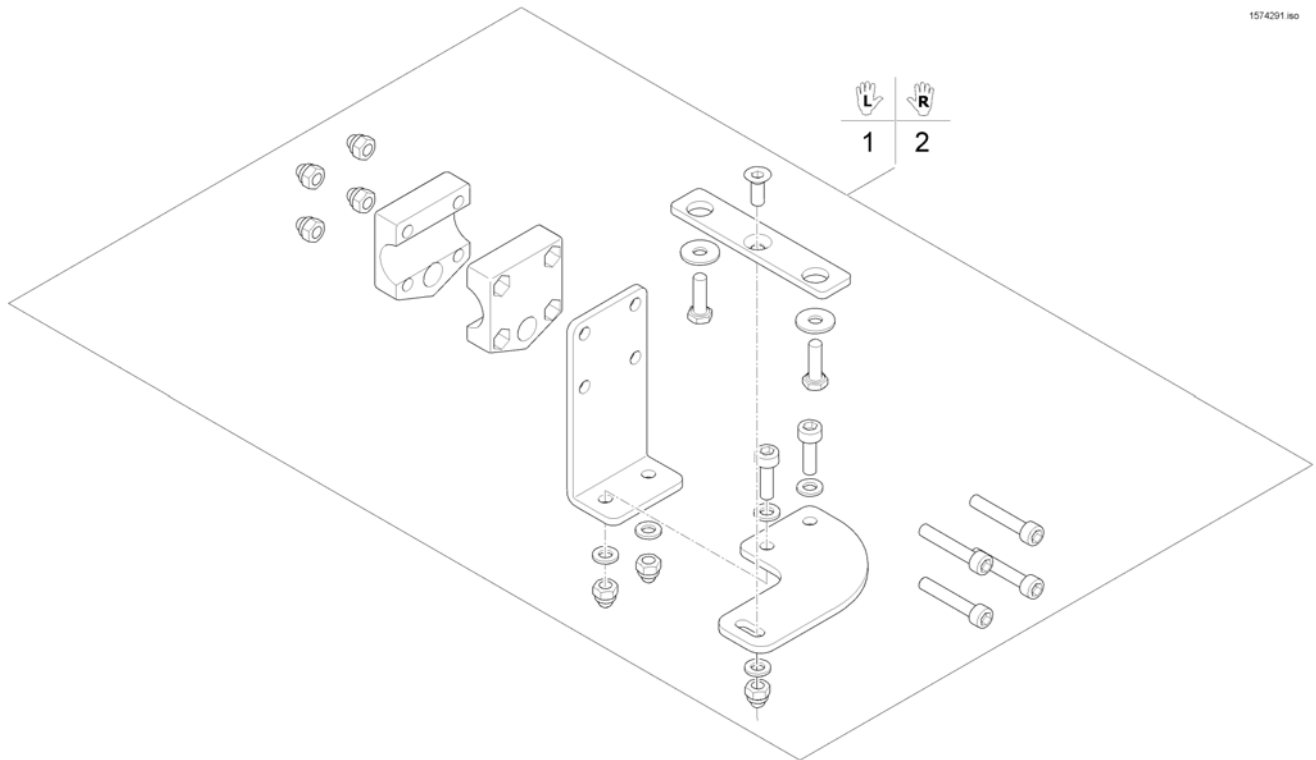

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571799	Remote Bracket Dual Control LH		1
2	SP1571800	Remote Bracket Dual Control RH		1

Compact Dual Control with brackets

1588802 Rev

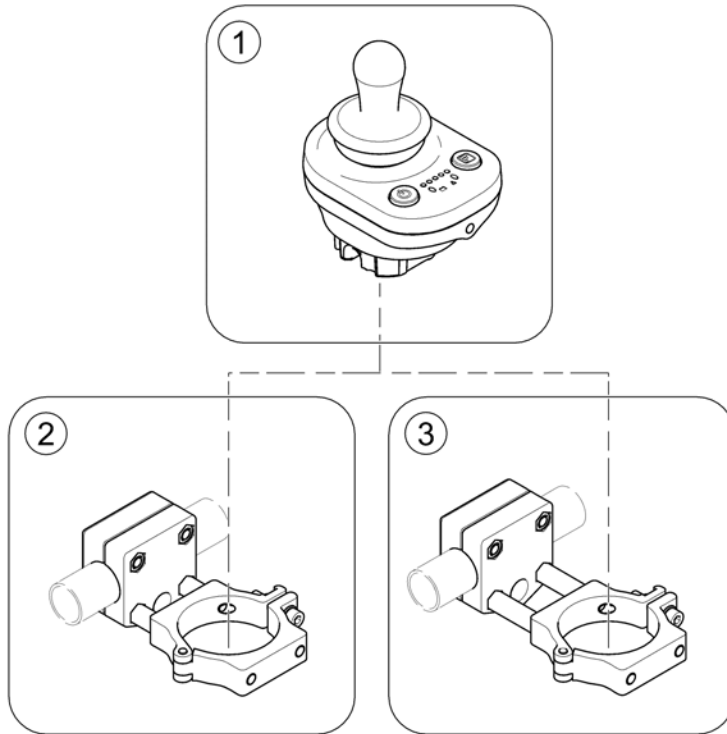

Pos.	Part number	Description	Valid from → until	Qty
1	-	Compact Dual Control		1
2	-	Bracket, Compact Dual Control - short		1
3	-	Bracket, Compact Dual Control - long		1

Compact Dual Control


1586988.jpg



Pos.	Part number	Description	Valid from → until	Qty
1	SP1586988	Remote DX2-ACU		1

Bracket, Compact Dual Control - short


1588857 .ind

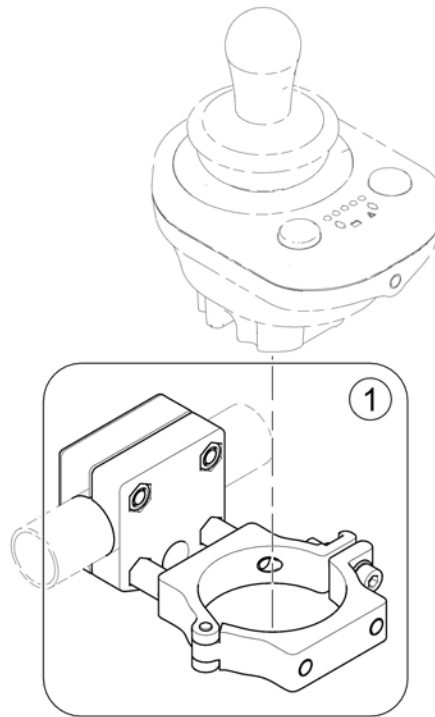

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587904	Remote Attachment - short		1

Bracket, Compact Dual Control - long


158856.iss

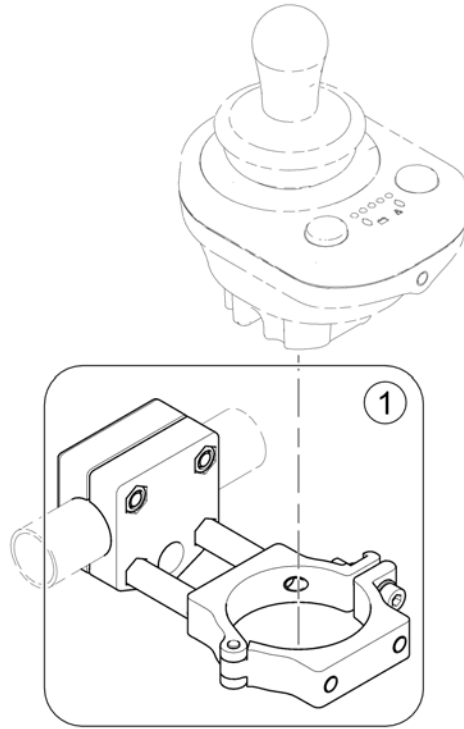

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587119	Remote Attachment - long		1

Intuitive Dual Control with brackets

1509875X.iso

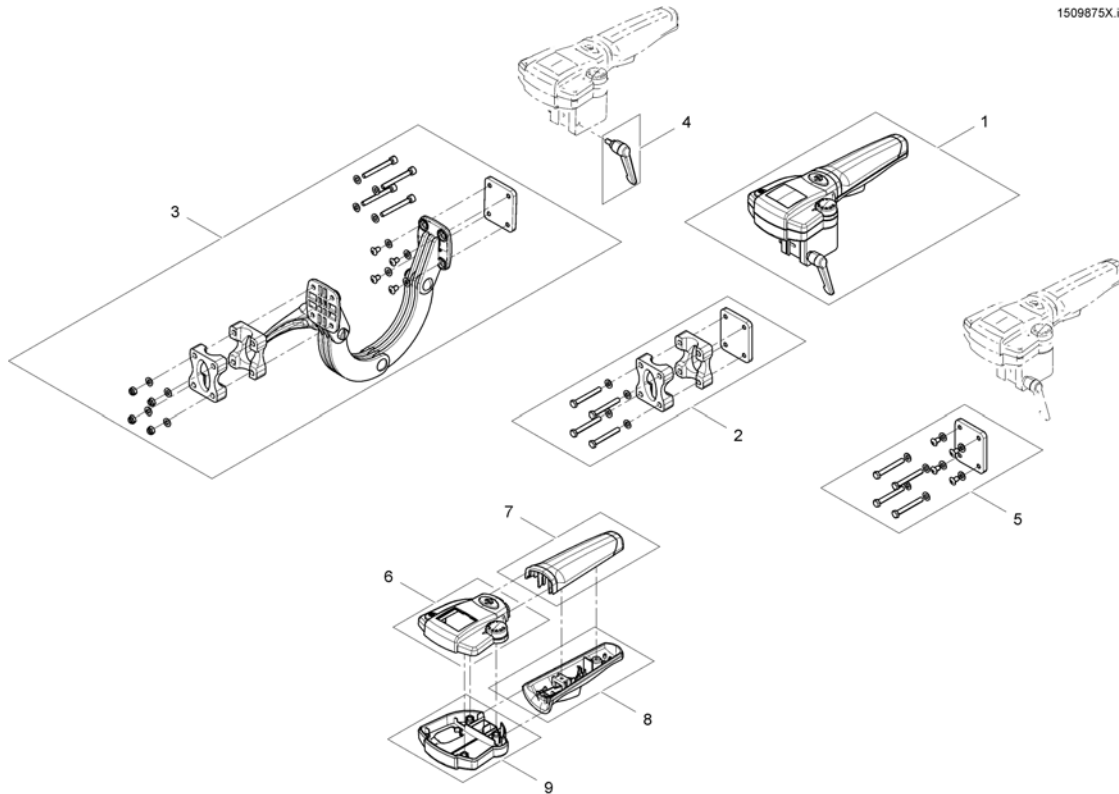


Pos.	Part number	Description	Valid from → until	Qty
1	SP1509875	Intuitiv Dual Control (IDC) <i>For chairs build before 01/02/10 please also order SP1537148</i>		1
2	SP1522310	Fixed bracket		1
3	SP1521488	Adjustable bracket		1
4	SP1510135	Clamp lever M5x10		1
5	SP1537148	Supportplate IDC for chairs before 01.02.10		1
6	SP1579573	Upper Housing		1
7	SP1511060	Handhold upper part		1
8	SP1511099	Handhold lower part		1
9	SP1510470	Housing lower part		1

ACS Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables ACS		1
-		T-Adapter ACS 2		1
-		T-Adapter - Modulite Seat		1
-		Slow-Stop-Sensor & Holder		1
-		Programming Kits		1
-		Magnetkey for Remotes		1


ACSBUSKABEL.iso

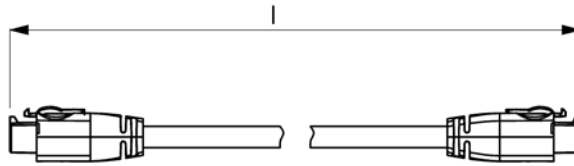

Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

Pos.	Part number	Description	Valid from → until	Qty
1	GSM63003	Bus cable ACS, 300mm		1
2	SPGSM63005	Bus cable ACS, 500mm <i>Replace GSM63004</i>		1
3	GSM63007	Bus cable ACS 700mm		1
4	GSM63011	Bus cable ACS, 1100mm		1
5	1559862	Bus cable ACS, 1700mm <i>Replace GSM63016</i>		1
6	GSM63020	Bus cable ACS, 2000mm <i>Replace 1419386</i>		1
7	1440424	Bus cable ACS, 2500mm		1
8	SP1559873	Bus cable ACS, 2700mm <i>Replace GSM63026</i>		1
9	SP1550402	Extension cable ACS, 1060mm		1


T-Adapter ACS 2

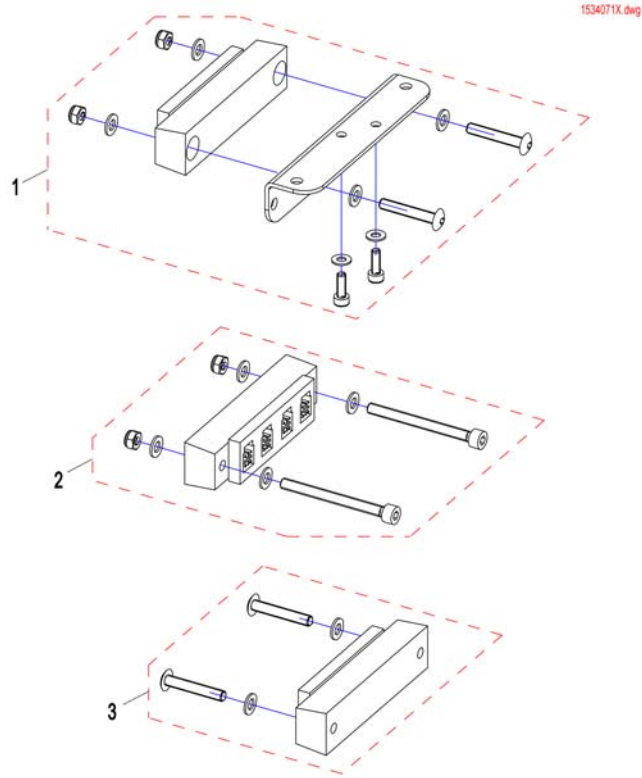

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536863	T-Adapter ACS 2 for Flex2 with lifter		1
2	SP1534948	T-Adapter ACS 2 for Firmseat with / without lifter and Flex2 without lifter		1
3	SP1534072	T-Adapter ACS 2 for Standardseat with / without lifter		1

T-Adapter - Modulite Seat


1583360 iso

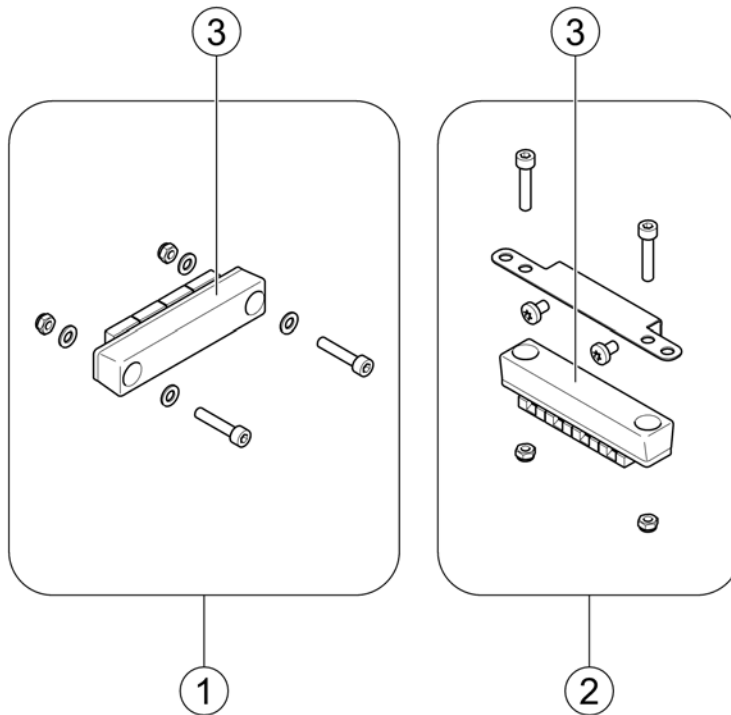


Pos.	Part number	Description	Valid from → until	Qty
1	SP1571826	T-Adapter - Modulite Seat		1
2	SP1569958	T-Adapter - Modulite Seat with MBL-Modul		1
3	P10072	ACS-Bus T-Adapter (Splitter)		1

Slow-Stop-Sensor & Holder

1533662X.dwg

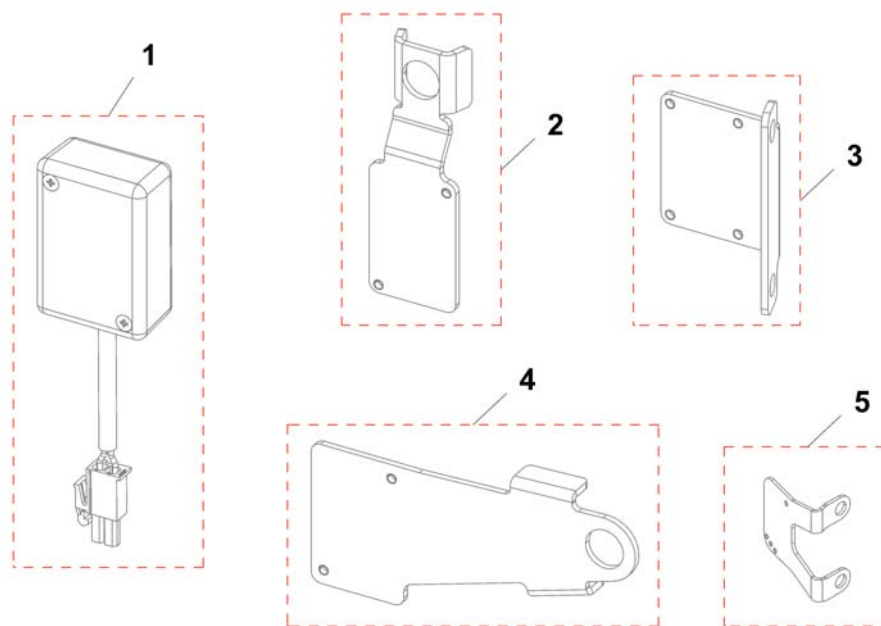


Pos.	Part number	Description	Valid from → until	Qty
1	SP1533662	Slow-stop-sensor, ACS2, cpl.		1
1	SP1572175	Slow-stop-sensor, ACS2, Modulite, cpl.		1
1	SP1537477	Slow-stop-sensor, R-Net, cpl.		1
1	SP1571202	Slow-stop-sensor, R-Net, Modulite, cpl.		1
2	SP1533665	Holder slow-stop-sensor Standardseat		1
3	SP1533661	Holder slow-stop-sensor Firm-seat		1
4	SP1534049	Holder slow-stop-sensor Flex2		1
5	SP1563880	Holder slow-stop-sensor Modulite		1

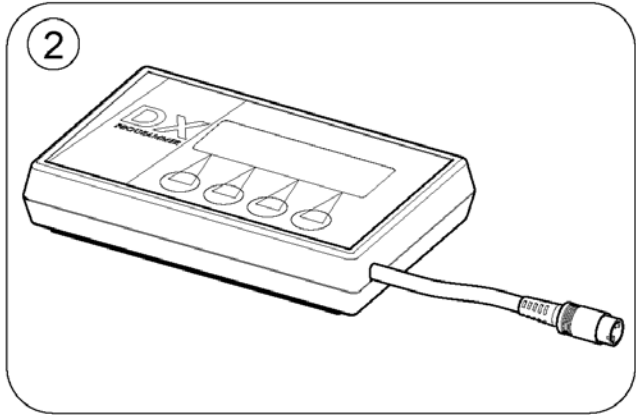

Pos.	Part number	Description	Valid from → until	Qty
1	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	6001450	Hand Programmer, Instrucions In French, German, Spanish		1
	SP1441551	Programming Adapter Shark DK-ADAPT (necessary to program Shark)		1

Magnetkey for Remotes

1564186.iso

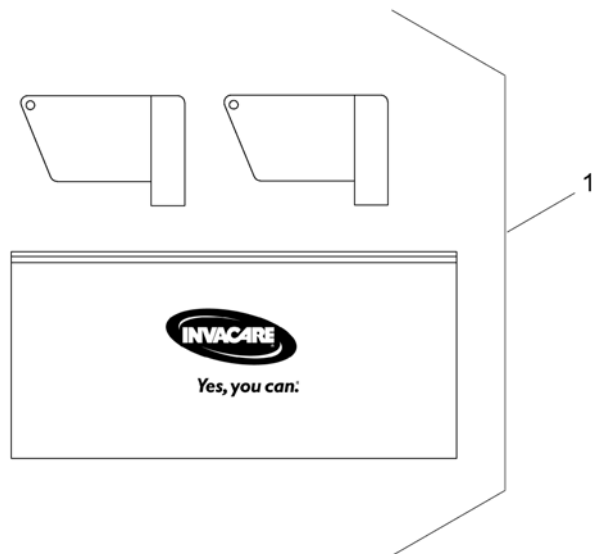

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419695	Magnet key with bag		1

Batteries


	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1													14		
Storm3 Vertic														14		
Storm4														14		
Storm4 Max														14		
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow												12				
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion												11				
length [mm]	97	152	197	196	211	196	210	197	198	198	198	238	259	260	260	329
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5



Pos.	Part number	Description	Valid from → until	Qty
i	-	Change batteries only pairwise and from the same type!		1
1	1440386	Battery, 12V/1,2Ah C20, GEL		1
2	SP1506493	Battery 12V/12Ah AGM		1
3	P50021	Battery, 12V/30Ah C20, GEL		1
4	P50029	Battery, 12V/31Ah C20, GEL		1
5	SP1521459	Battery, 12V/34Ah C20, AGM		1
6	1505498	Battery, 12V/36Ah C20, AGM		1
7	P50028	Battery, 12V/38Ah C20, GEL (UK & DK only)		1
8	PG50988	Battery, 12V/40Ah C20, GEL		1
9	SP1550692	Battery, 12V/40Ah C20, AGM		1
10	SP1522107	Battery, 12V/50Ah C20, AGM		1
11	SP1521695	Battery, 12V/50Ah C20 AGM		1
12	PG50989	Battery, 12V/50,9Ah C20, GEL		1
13	1432944	Battery, 12V/60Ah C20, GEL		1
14	P50016	Battery, 12V/73,6 Ah C20, Gel		1
15	SP1522026	Battery, 12V/75Ah C20, AGM		1
16	1478675	Battery, 12V/97,6Ah C20, GEL		1

Battery accessories


1546904X.dwg

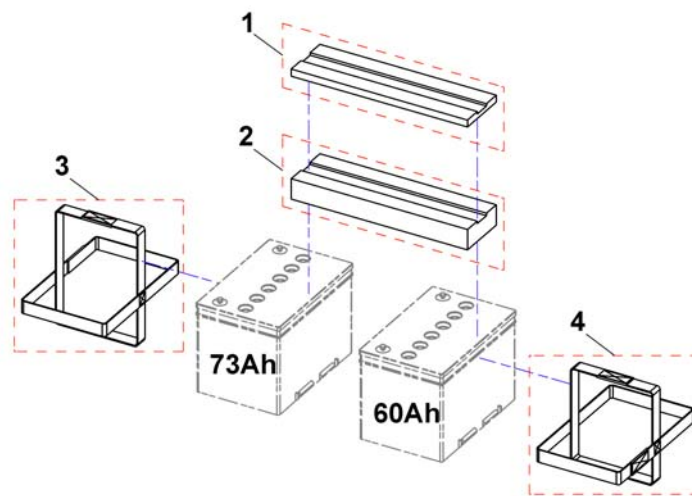

Pos.	Part number	Description	Valid from → until	Qty
1	SP1546565	Spacer for 73Ah batteries		1
2	SP1546566	Spacer for 60Ah batteries		1
3	SP1546718	Carrying belt for 73Ah batteries		1
4	SP1546719	Carrying belt for 60Ah batteries		1

Battery cable ACS 2


1547844_002

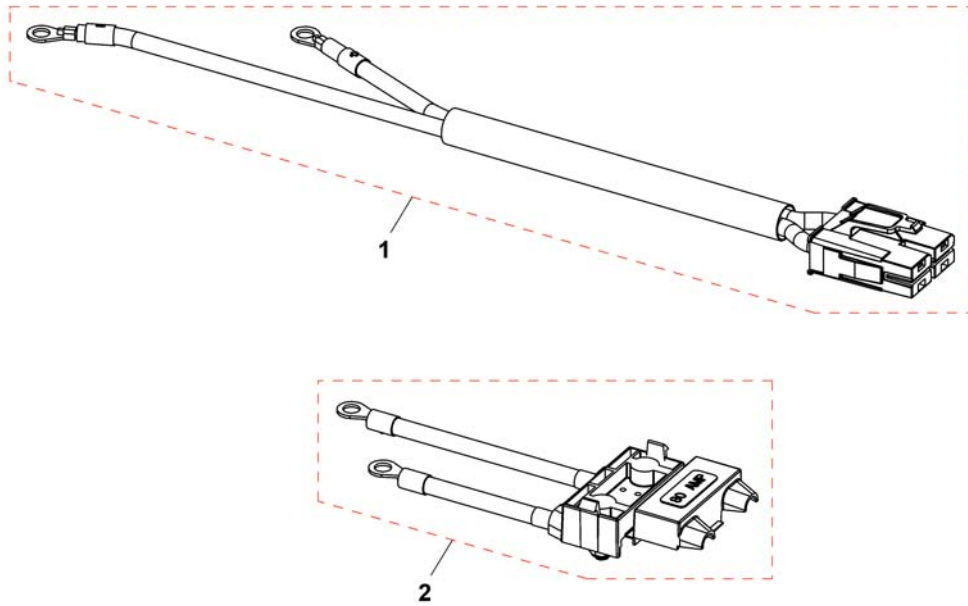

Pos.	Part number	Description	Valid from → until	Qty
1	SP1546421	Batterycable ACS 2		1
2	SP1546422	Fusecable		1
-	SP1588075	Cable Tie Set V0 (10 of each T18R/T30R/T50R)		1

Chargers


1589799.iso

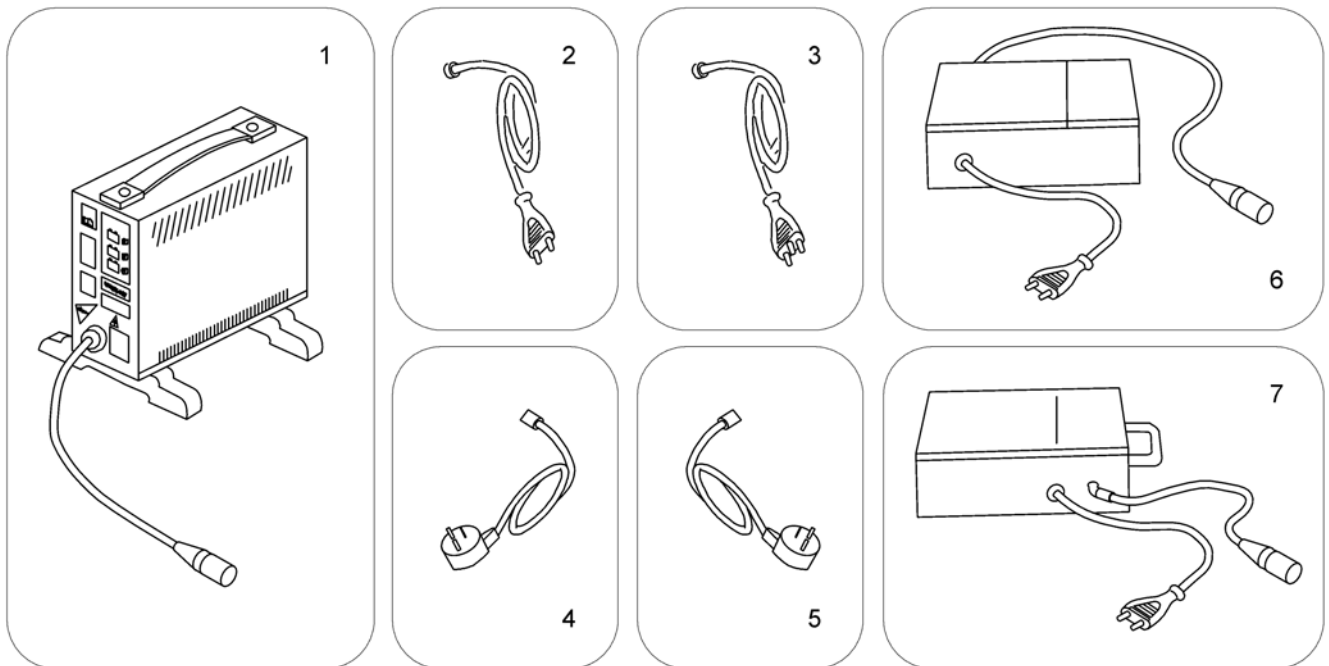

Pos.	Part number	Description	Valid from → until	Qty
1	SPP51981	Charger 8A without Power Cord <i>[1343]</i>		1
2	P21079	Power Cord Continental (Schuko GCB-0401)		1
3	P21080	Power Cord UK (GCB-0403)		1
4	1443841	Power Cord Swiss (black 6A/250V)		1
5	SP1553849	Power Cord Israel (IEC 320 C13 to SI-32)		1
6	SP1545511	Charger 10A incl. Power Cord Continental <i>[1389]</i>		1
7	5813987	Charger 8A (Protectionclass IP54) incl. Power Cord Continental (Sweden only) <i>[1388]</i>		1

Invacare® Kite | Safety Options & Accessories
Frontlight (German homologation)


1548873-04q

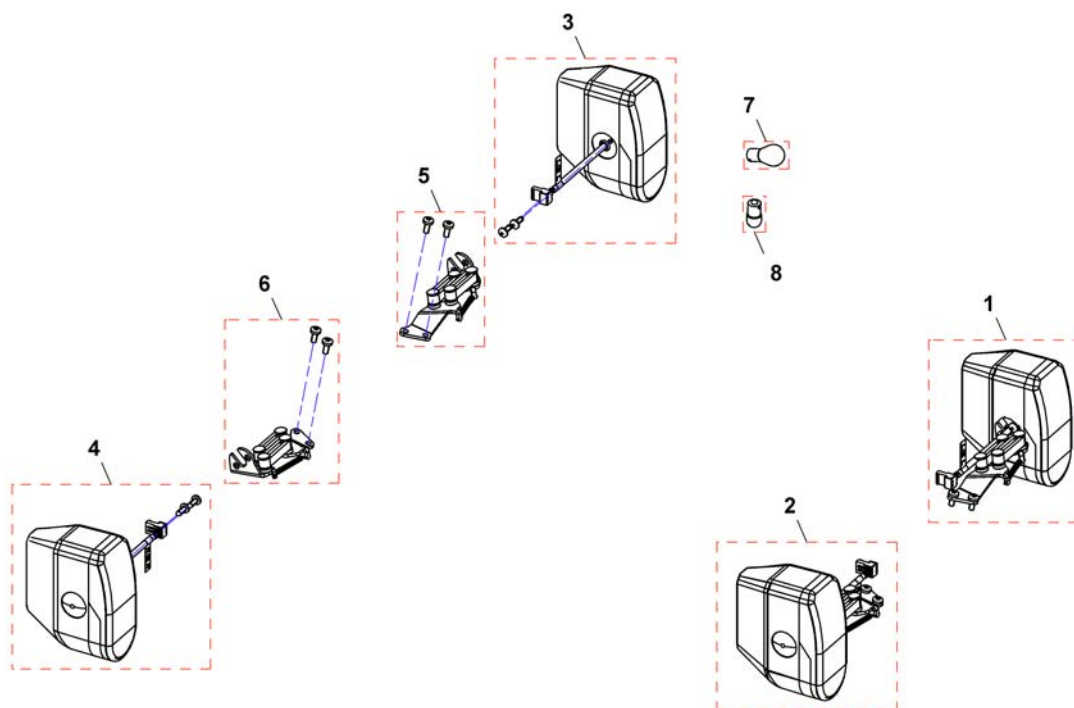

Pos.	Part number	Description	Valid from → until	Qty
1	SP1550704	Frontlight, cpl., LH		1
2	SP1550705	Frontlight, cpl., RH		1
3	SP1550822	Headlamp, LH		1
4	SP1550823	Headlamp, RH		1
5	SP1550824	Headlight bracket, cpl., LH		1
6	SP1550825	Headlight bracket, cpl., RH		1
7	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
8	SPP31108	Light Bulb 24V/10W (BA15s), white (Front)		1

Rearlight (German homologation)


1548707K.dwg

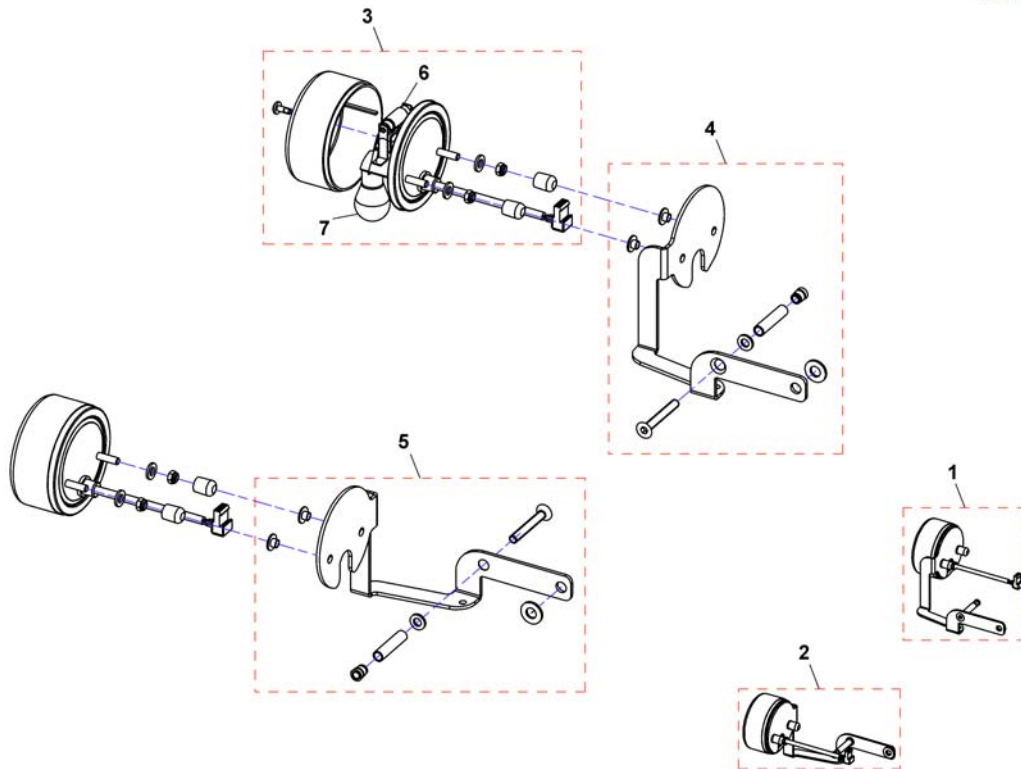

Pos.	Part number	Description	Valid from → until	Qty
!!!	-	Rear Light Protector Kit		0
1	SP1548707	Rearlight with holder, LH, cpl.		1
2	SP1548706	Rearlight with holder, RH, cpl.		1
3	SP1550826	Rearlight		1
4	SP1548704	Holder for rearlight, LH		1
5	SP1548703	Holder for rearlight, RH		1
6	P31191	Light Bulb 24V/5W, white (Rear)		1
7	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1

Rear Light Protector Kit


1585420 (6)

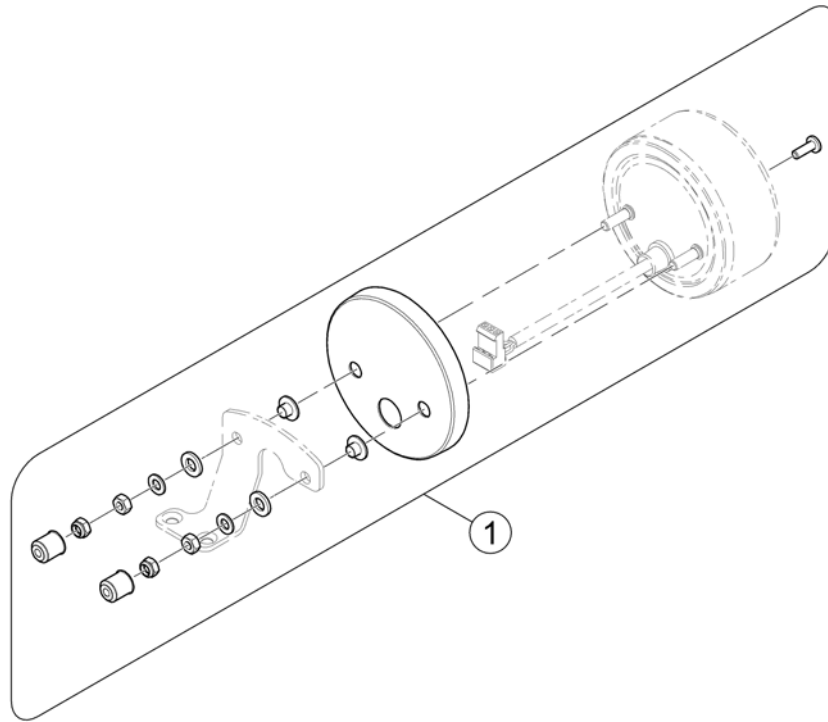

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581827	Rear Light Protector Exchange Kit		1

LED-Frontlight


162026-04g

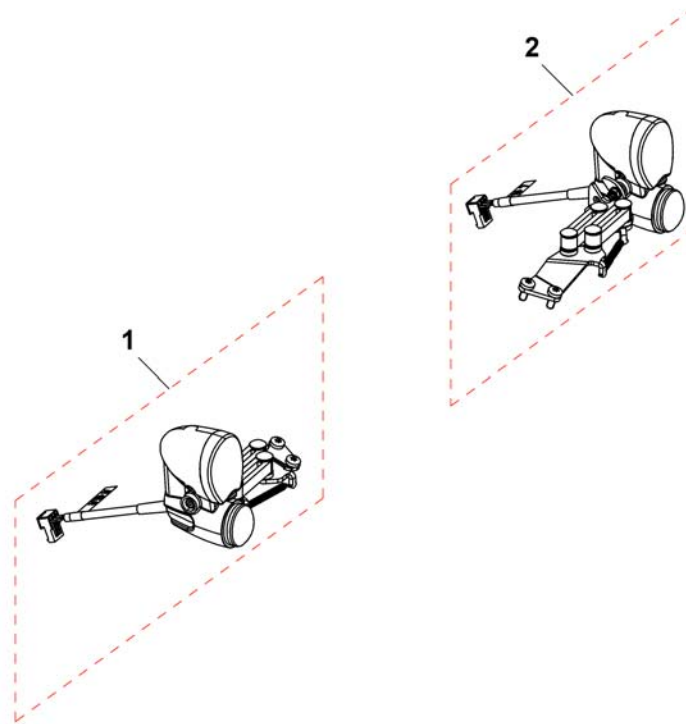

Pos.	Part number	Description	Valid from → until	Qty
1	SP1552489	Frontlight, LED, RH		1
2	SP1552437	Frontlight, LED, LH		1

LED-Rearlight


1551403X.dwg

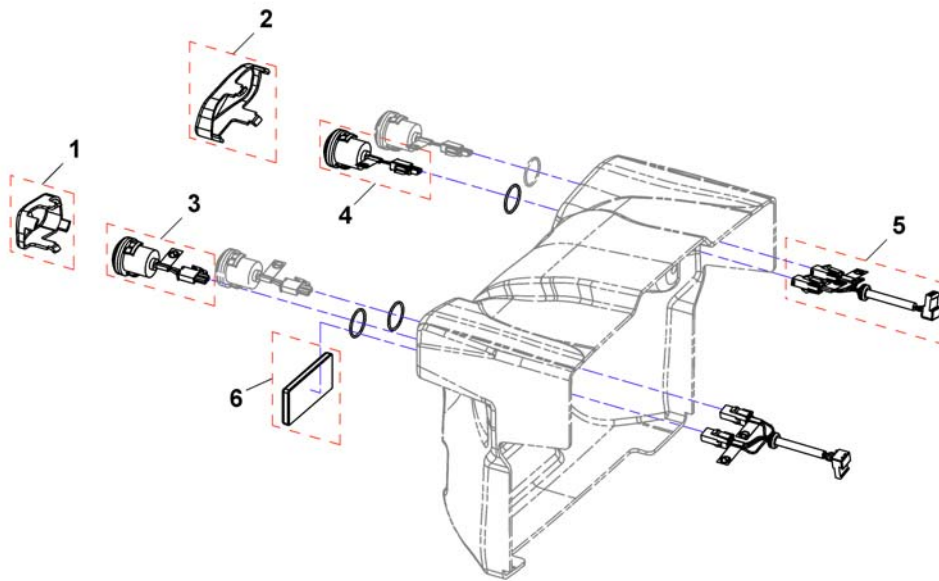

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551208	Lampglass RH		1
2	SP1551209	Lampglass LH		1
3	SP1530287	Turn indicator		1
4	SP1530288	Rear light		1
5	SP1529712	Rear light LED-Cable		1
6	5807542	Reflector angled, red		1

Reflective Sign


SP1430410.iss

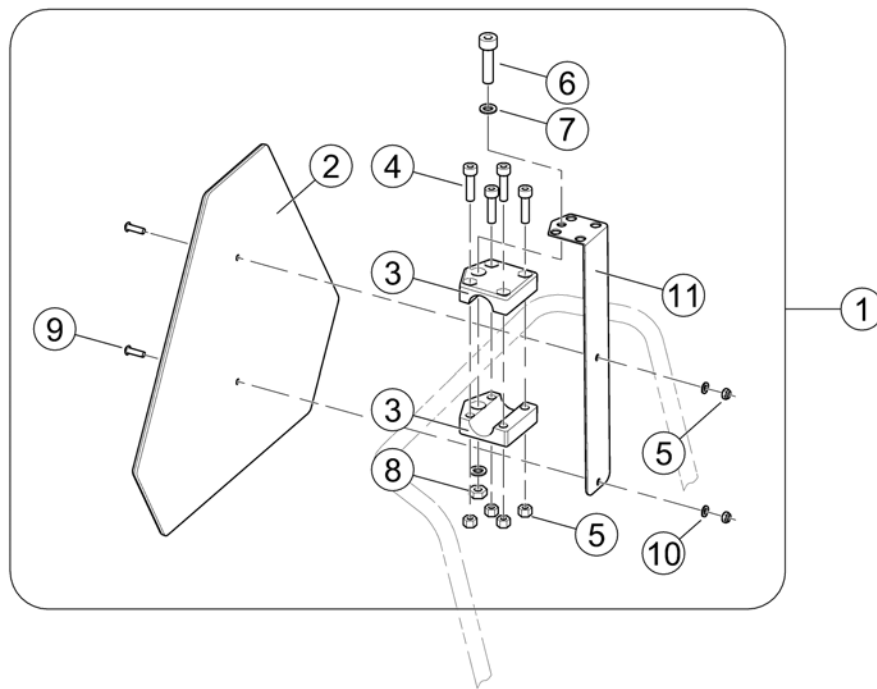

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430410	Reflective Sign ECE-R 69, complete		1
2	1430834	Rear Marking Sign ECE-R 69		1
3	Z12008A.P40	Gripping clip with hexagon hole, RAL 9005 black		2
4	D50004	Cylinder Bolt M6x30-8.8 bk - DIN 912		4
5	F20109	Hexagon Nut M6 bk - DIN 985		6
6	D00102	Hexagon Bolt M8x45-8.8 bk - DIN 931		1
7	G00107	Washer Ø8,4 bk - DIN 125		2
8	F00002	Hexagon Nut M8 bk - DIN 985		1
9	D90187	Fillister Head Screw M6x20-10.9 bk - DIN 7380		2
10	G00115	Washer Ø6,4 bk - DIN 125		2
11	1430418-9005	Mounting Bracket for Reflective Sign - RAL 9005 black		1
12	SP1580671	Bracket, Reflective Sign, cpl.		1


Kerb Rider

1544305X.dwg

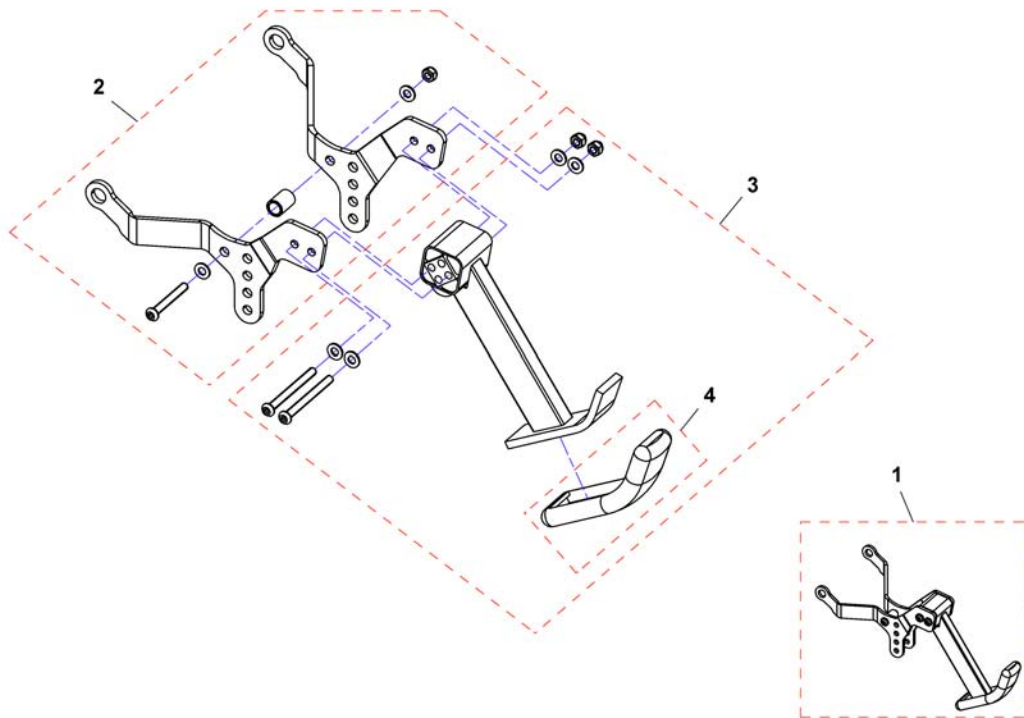

Pos.	Part number	Description	Valid from → until	Qty
1	SP1544305	Mounting kit kerb climber, cpl.		1
2	SP1550702	Tilt mounting hardware to adapt kerb climber		1
3	SP1550703	Kerb climber		1
4	B11037B	Rubber Foot		1


Lap Belt

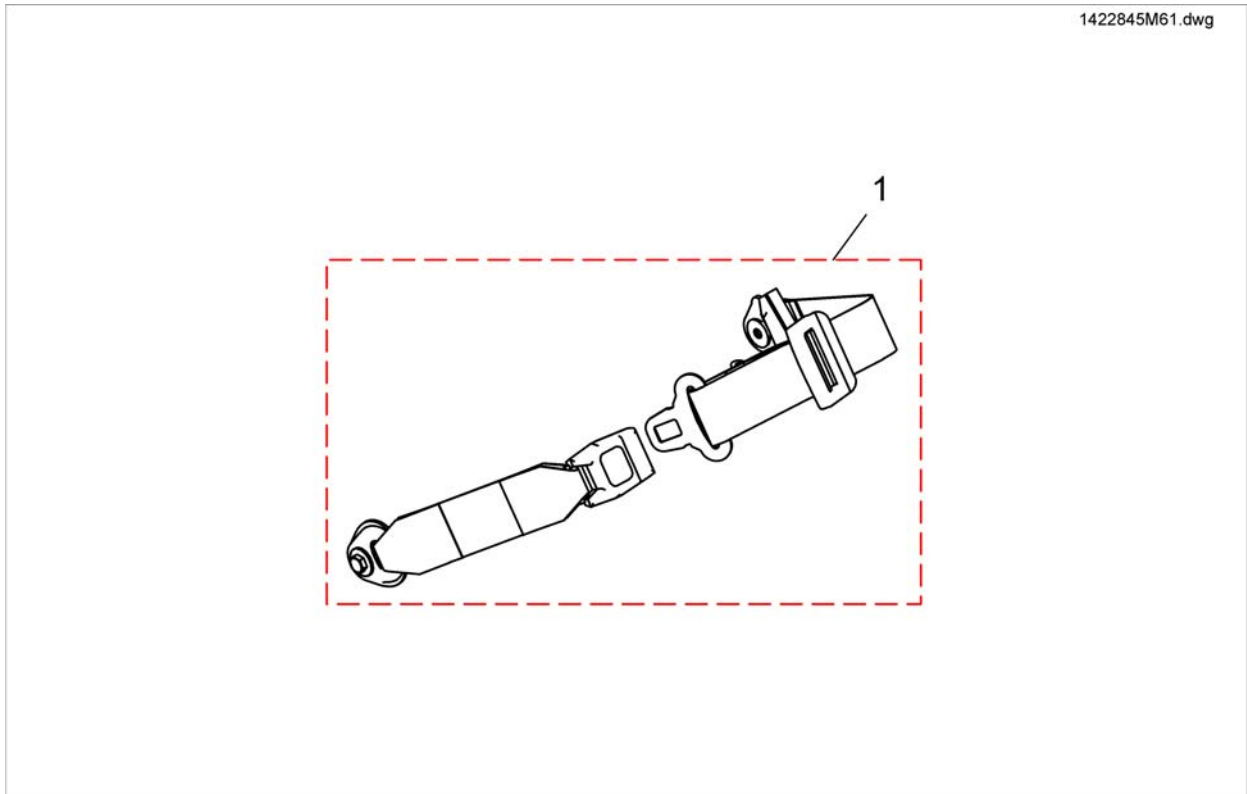

Pos.	Part number	Description	Valid from → until	Qty
1	1422845	Lap Belt, cpl.		1


Lap Belts (Modulite Seat)

1571836 iso

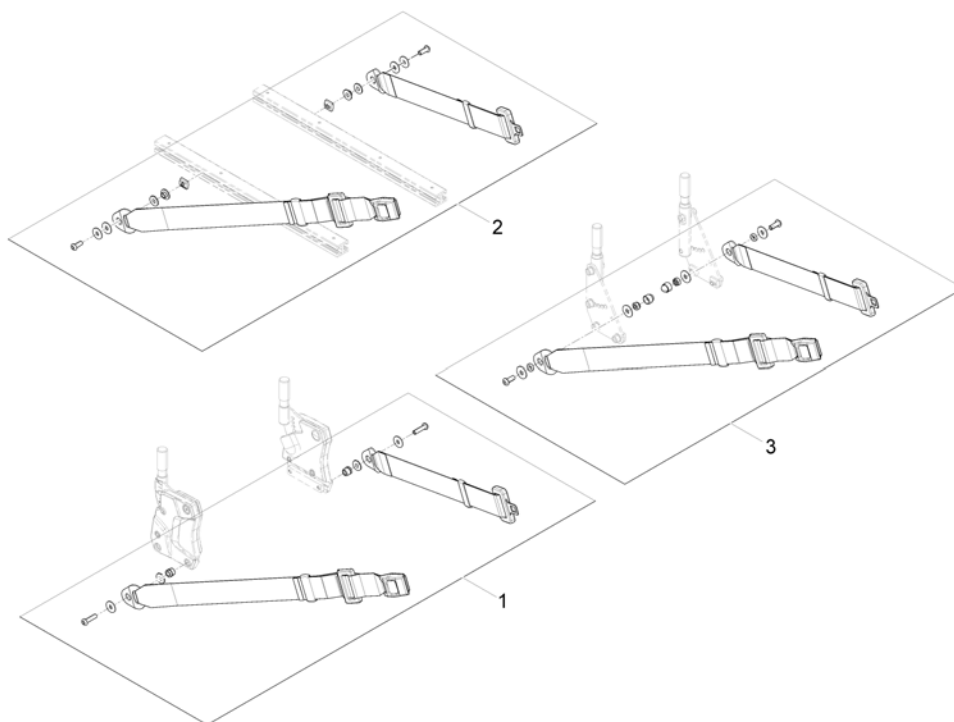

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570381	Lap Belt (fix mounting on Backrest Support), cpl.		1
2	SP1571183	Lap Belt (flexible mounting in side rails), cpl.		1
3	1422845	Lap Belt, cpl.		1

Rearview Mirror

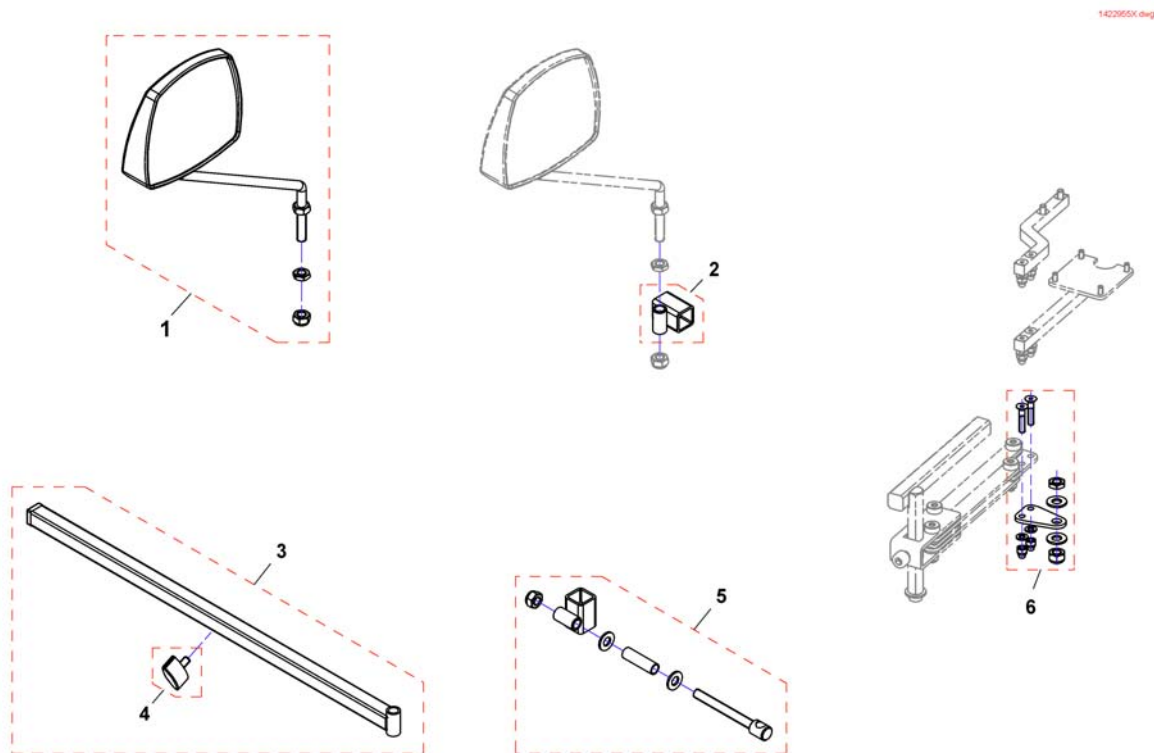


Pos.	Part number	Description	Valid from → until	Qty
1	SP1422955	Rear Mirror		1
2	F276005.00	Mirror bracket		1
3	SP1423435	Rear Mirror Support Rail cpl.		1
4	L31001	Wing Scew M6X10		1
5	SP1422957	Mirror bracket		1
6	SP1527151	Adapter to mount mirror and remote at the same side		1

Lateral Support

Pos.	Part number	Description	Valid from → until	Qty
-		Lateral Support - Flex 2 Seat		1
-		Lateral Support - Modulite Seat		1

Lateral Support - Flex 2 Seat


Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral Support (Dartex) [1446]		1
2	-	Lateral Support, extended arm (Dartex) [1445]		1

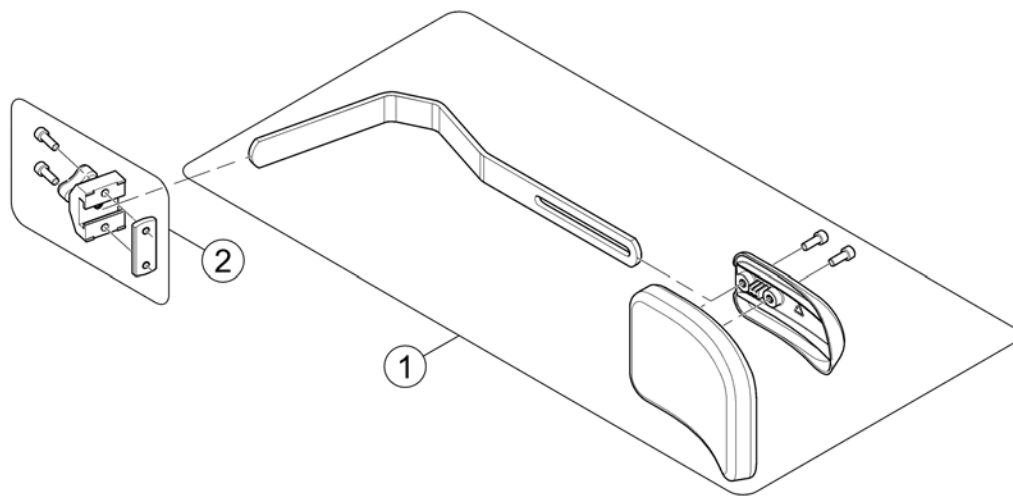

Pos.	Part number	Description	Valid from → until	Qty
1	SP1561841	Trunk support, cpl.		1
2	5253170	Lateral Support Bracket		1

Lateral Support, extended arm (Dartex)

1502674.iss

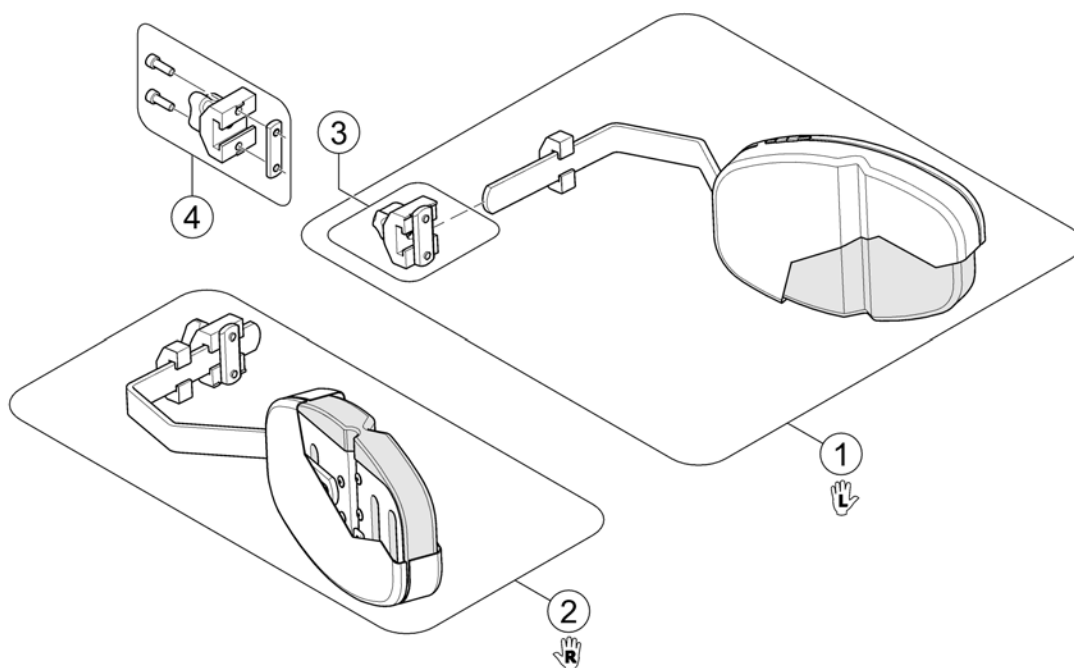


Pos.	Part number	Description	Valid from → until	Qty
1	1448692	Lateral Support, multi adjustable - RH <i>Dartex grey</i>		1
2	1448693	Lateral Support, multi adjustable - RH <i>Dartex grey</i>		1
3	5253170	Lateral Support Bracket <i>Only for Flex/Flex2 Seat!</i>		1
4	1543574	Locking trunk support <i>Only for Modulite Seat!</i>		1

Lateral Support - Modulite Seat

Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral Support (Dartex) [1446]		1
2	-	Lateral Support, extended arm (Dartex) [1445]		1

Lateral Support (Dartex)

158860 iso

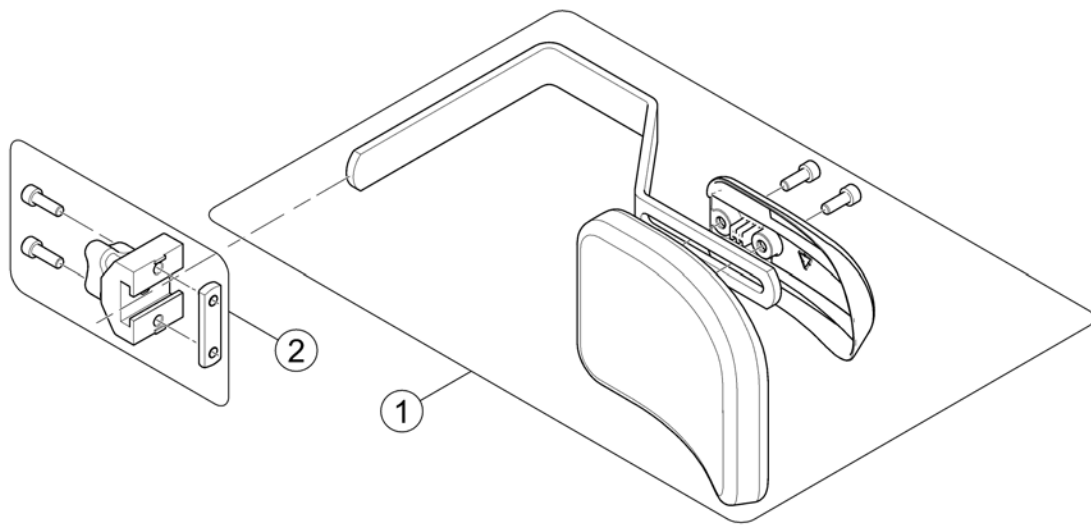


Pos.	Part number	Description	Valid from → until	Qty
-		Design change on 01. July 2015 - we suggest to change older Supports on both sides.		0
1	SP1588314	Lateral Support, cpl.		1
2	1543574	Locking trunk support		1

Headrest

Pos.	Part number	Description	Valid from → until	Qty
-		Head- / Neckrest [1350]		1
-		Headrest sidewise adjustable [1432]		1
-		Headrest with cheek support [1427]		1


Head- / Neckrest

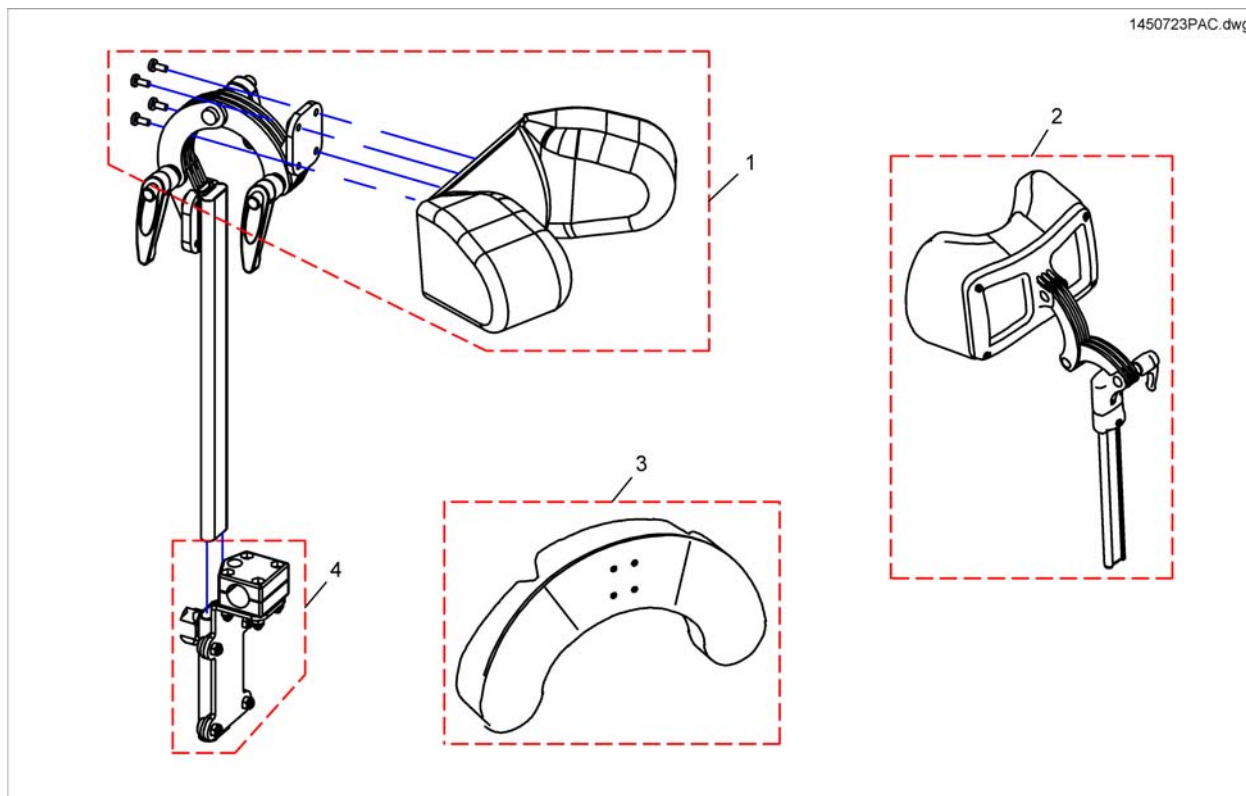

Pos.	Part number	Description	Valid from → until	Qty
1	1487404	Headrest cushion VCR, cpl.		1
2	SP1527050	Neckrest, depth adjustable		1
3	1444304	CUSHION, CHEEK SUPPORT		1
4	SP1422914	Head-/Neckrest Bracket, cpl.		1

Headrest sidwise adjustable


1497544.iso

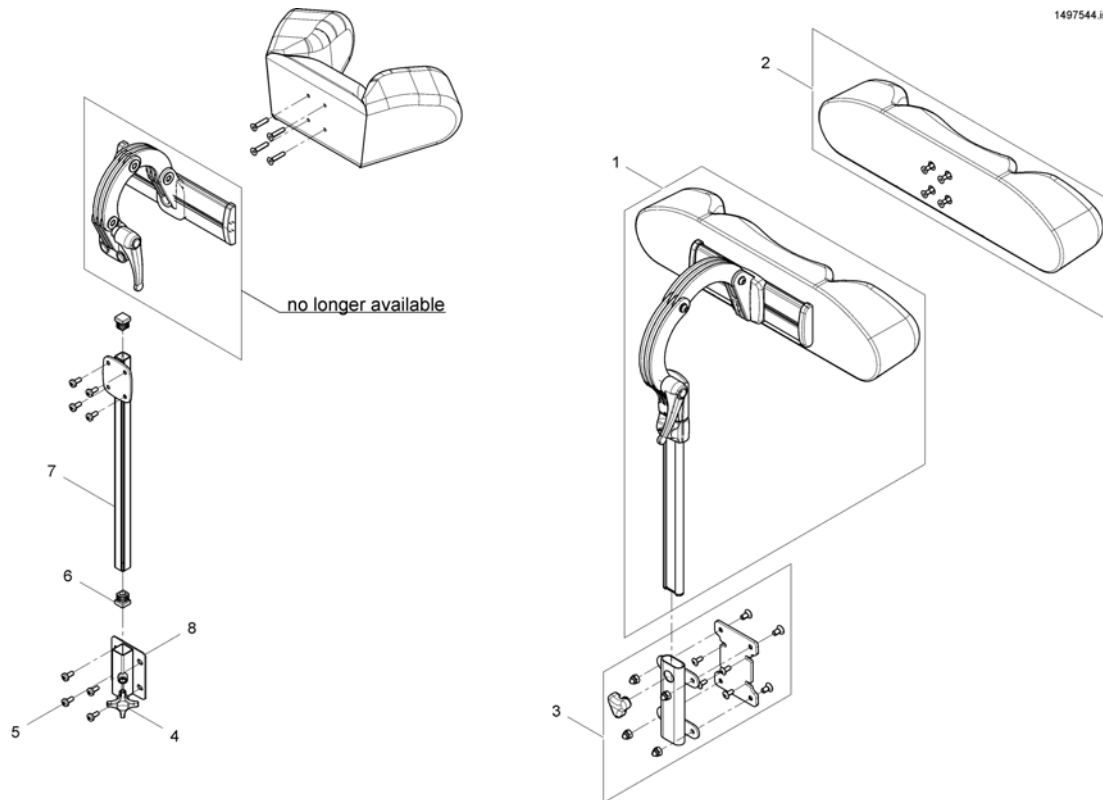

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530119	Headrest, sidwise adjustable, cpl. - Dartex TR26		1
2	SP1422865	Headrest cushion contour black, without bracket		1
2	1422866	HEADREST PAD STD./CONTOUR GREY		1
2	SP1522214	HEADREST PAD STD./CONTOUR BLACK PLUSH		1
3	SP1544680	MOUNTING-KIT		1
4	256099	Wing Screw M8 X 20 639		1
5	-	Lens head screw DIN7985-M05x12-4.8 zc		1
6	L01055	Plug 15x15mm		1
7	1451639-0205	Headrest REA Angle Adjustment		1
8	26002351	Tube Headrest		1

Headrest with cheek support


1446560X.dwg

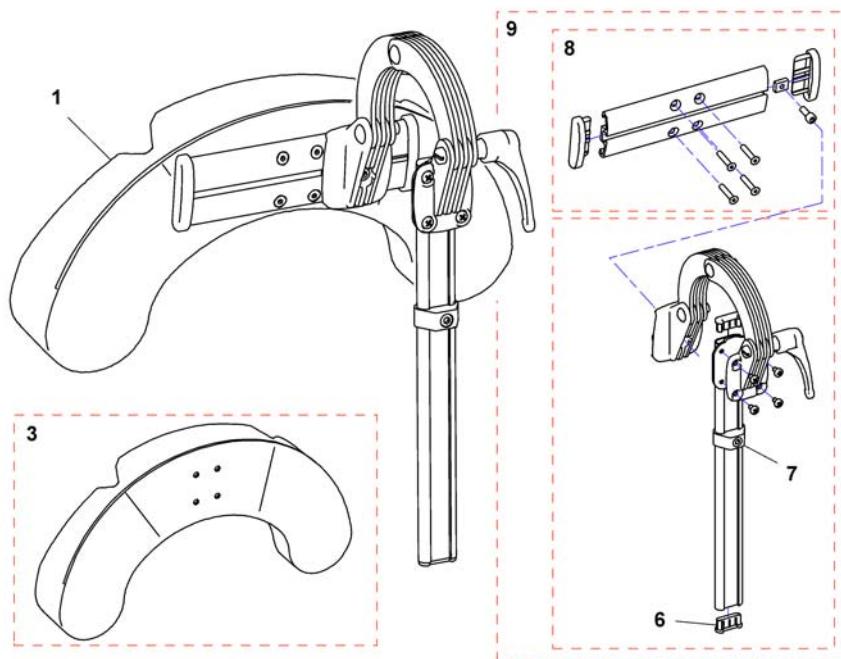

Pos.	Part number	Description	Valid from → until	Qty
1	1446560	Cheek Support, Sidewise Adj.		1
1	SP1541658	Head/cheek support TR26 cpl. sidewise adjustable EasyAdapt		1
3	1444304	CUSHION, CHEEK SUPPORT		1
5	SP1529572	Headrest holder for EasyAdapt		1
6	5716497	Plug IPO "special"		1
7	1448661	STOP CPL., NECK- & HEADREST		1
8	1445838	Rail head support		1
9	1485538	KIT, HEADREST ATTACHMENT		1

Abduction Cushion


1526960PAC.dwg

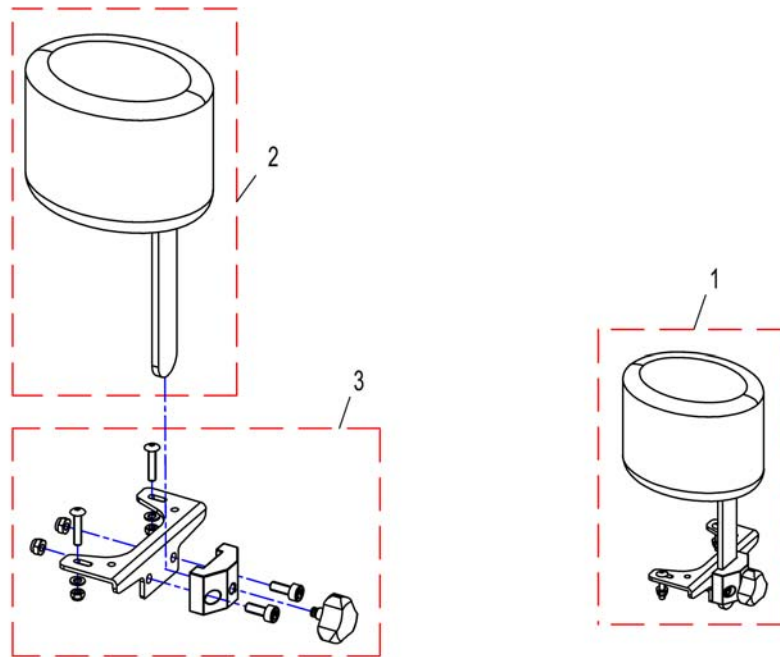

Pos.	Part number	Description	Valid from → until	Qty
1	SP1526960	Abduction Cushion. cpl.		1
2	1444172	POMMEL, DARTEX BLACK		1
3	1493099	Holder, cpl.		1

Luggage Rack

1551407X.dwg

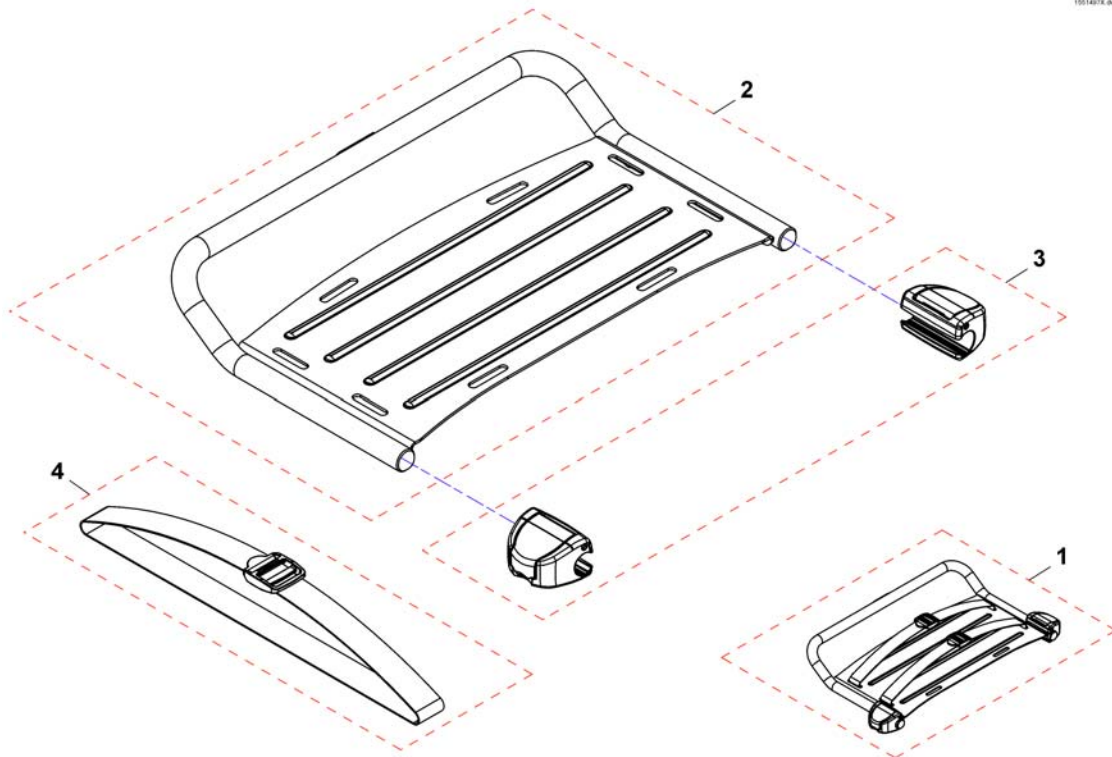


Only for chairs without lights, or with LED lights.

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551619	Luggage rack cpl.		1
2	SP1551497	Luggage rack		1
3	SP1556240	Luggage rack bracket		1
4	F384020	Lashing strap		1

Mounting Kit Luggage Rack


15518186.dwg

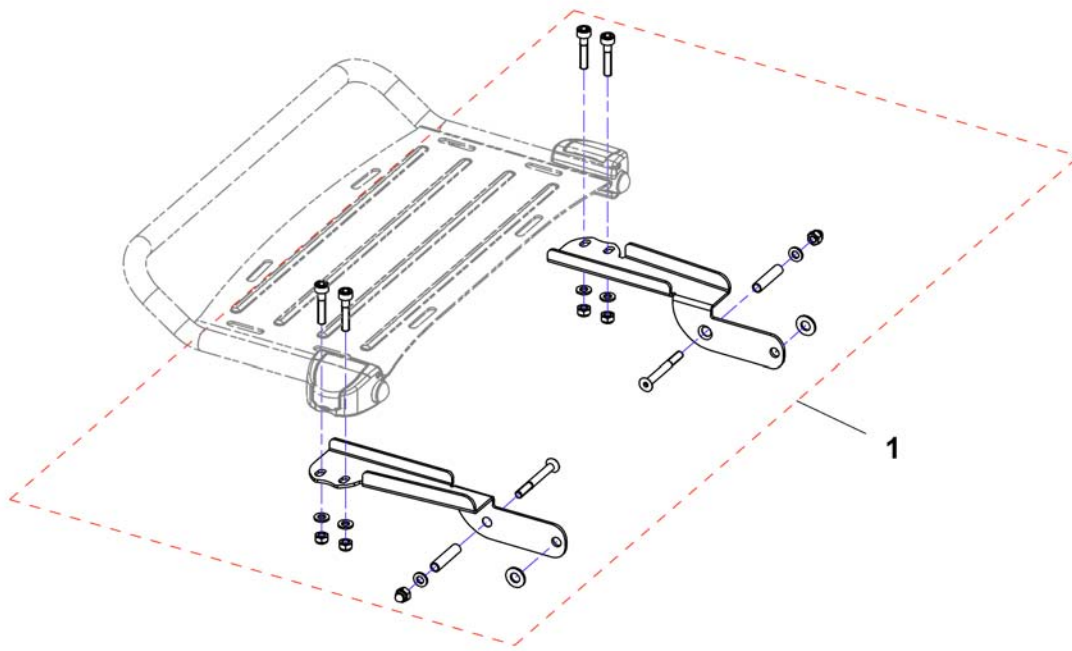

Pos.	Part number	Description	Valid from → until	Qty
1	SP1556242	mounting kit luggage		1


Cane holder (Std., Flex2, Max Seat)

1567253.iso

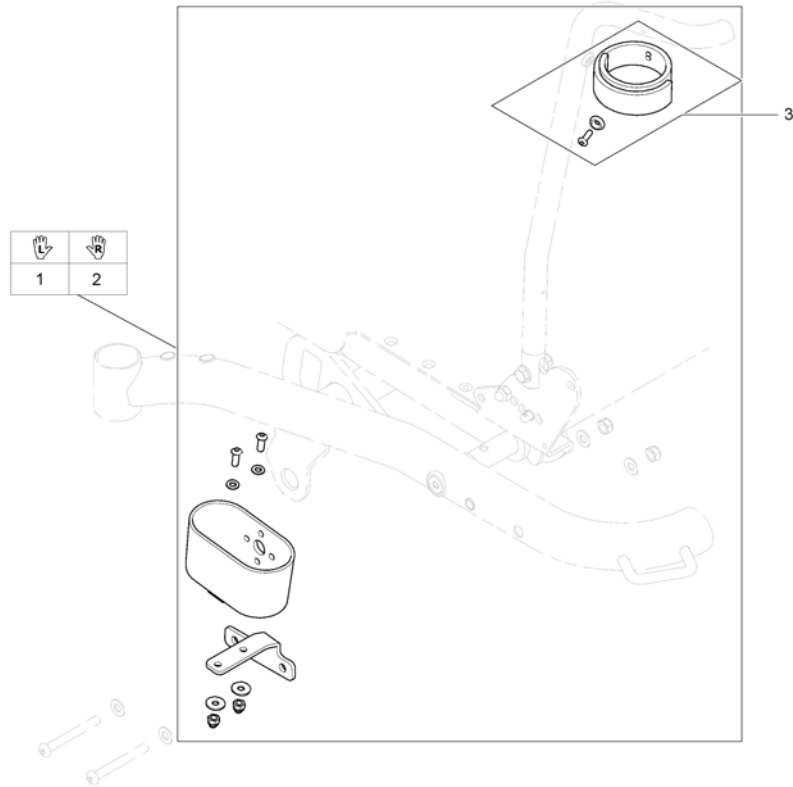


Pos.	Part number	Description	Valid from → until	Qty
1	SP1528488	Cane Holder cpl		1
2	SP1531230	Cane Holder cpl		1
3	SP1420493	Belt for caneholder		1

Cane holder (Modulite: backrest frame)

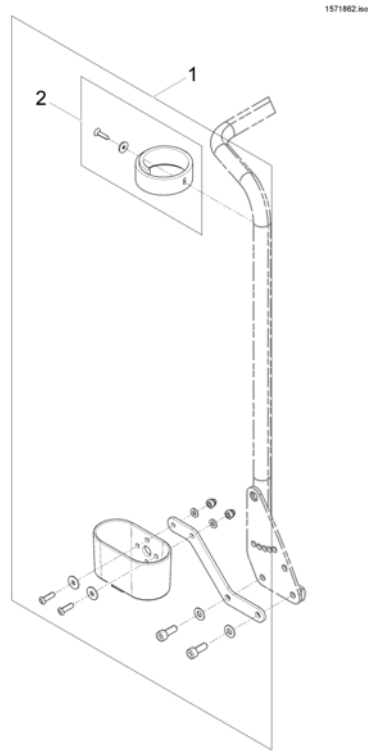

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570396	Cane holder, standard backrest		1
2	SP1420493	Belt for caneholder		1


Cane holder (Modulite: back posts man recliner)

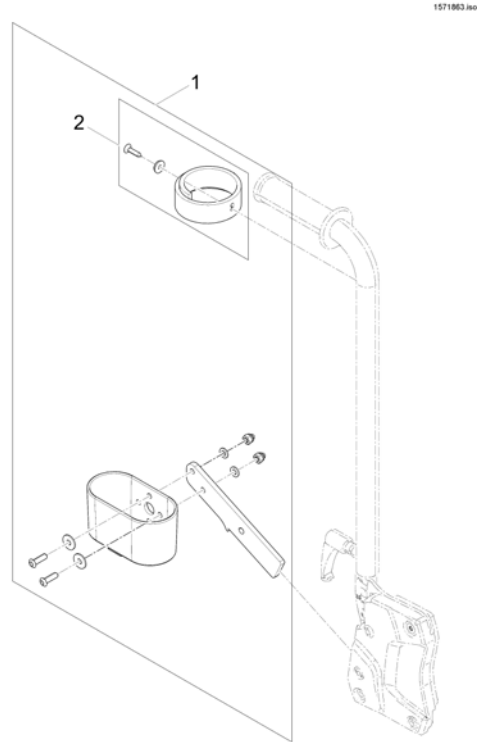

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570443	Cane holder, manual backrest		1
2	SP1420493	Belt for caneholder		1

Cane holder (Modulite: back posts pow recliner)


1571837.mo

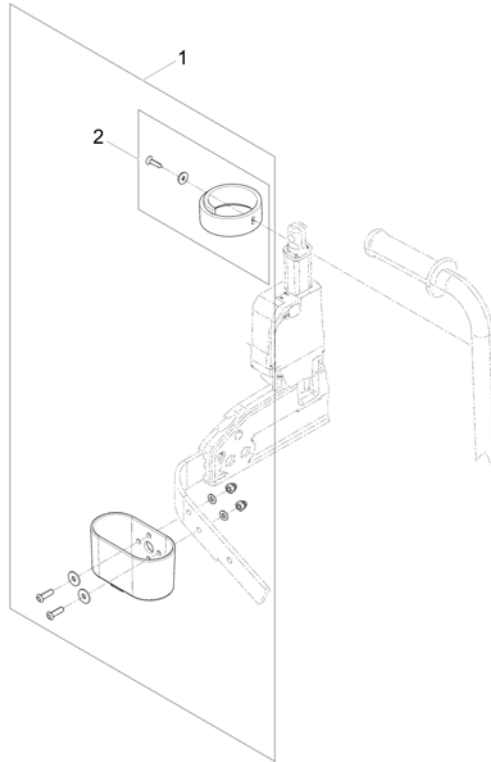

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570382	Cane holder, powered backrest		1
2	SP1420493	Belt for caneholder		1


Fender for Castor

1550699X.dwg

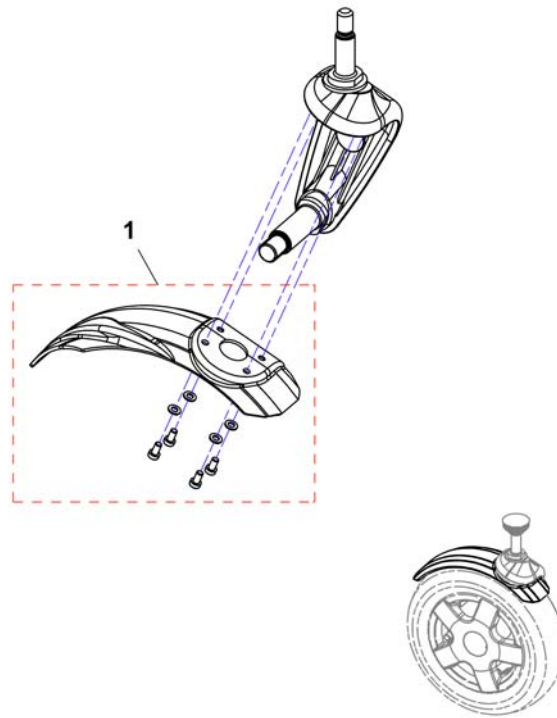


Pos.	Part number	Description	Valid from → until	Qty
1	SP1550699	Mudguard, cpl., LH		1
1	SP1550701	Mudguard, cpl., RH		1

1546906X.dwg

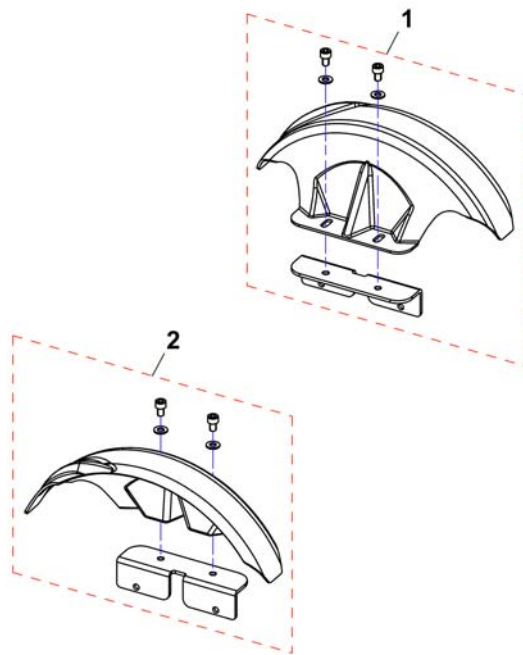

Pos.	Part number	Description	Valid from → until	Qty
1	SP1546321	Mud guard, cpl., LH		1
2	SP1546320	Mud guard, cpl., RH		1

Trays

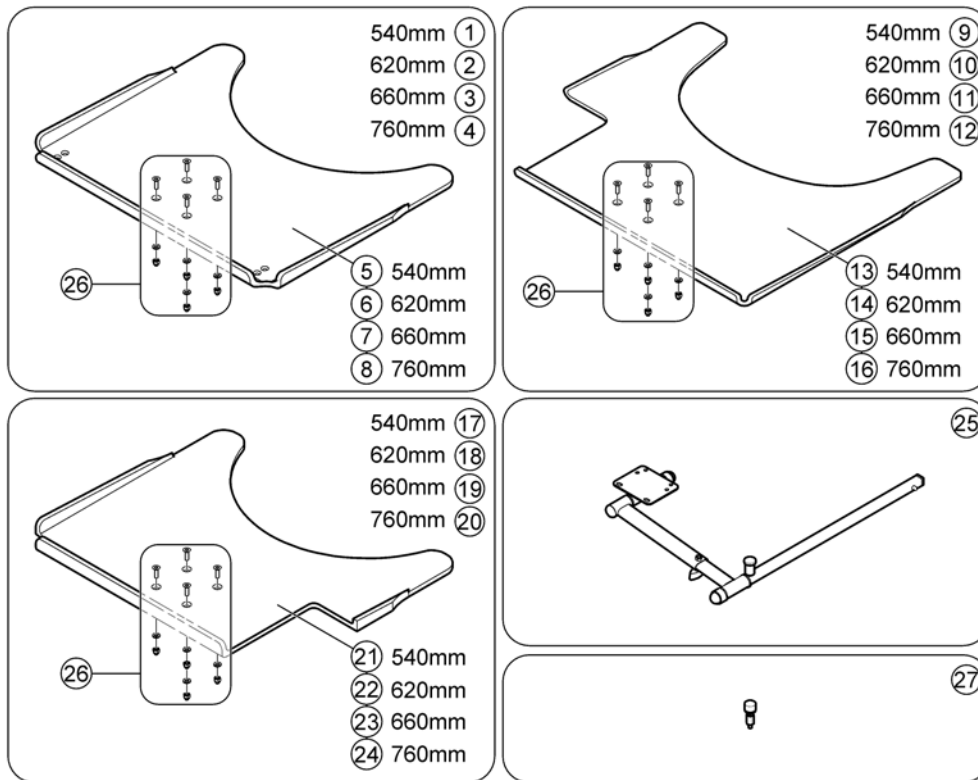


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548198	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, no Cutout, cpl. with holder		1
2	SP1548201	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, no Cutout, cpl. with holder		1
3	SP1422954	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, no Cutout, cpl. with holder		1
4	SP1584644	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, no Cutout, cpl. with holder		1
5	SP1535050	Tray table, 540mm, no Cutout		1
6	SP1535010	Tray table, 620mm, no Cutout		1
7	SP1535011	Tray table, 660mm, no Cutout		1
8	SP1586049	Tray table, 760mm, no Cutout		1
9	SP1548138	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout RH, cpl. with holder		1
10	SP1548199	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout RH, cpl. with holder		1
11	SP1422953	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout RH, cpl. with holder		1
12	SP1584663	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout RH, cpl. with holder		1
13	SP1535051	Tray table, 540mm, Cutout RH		1
14	SP1535012	Tray table, 620mm, Cutout RH		1
15	SP1535049	Tray table, 660mm, Cutout RH		1
16	SP1586048	Tray table, 760mm, Cutout RH		1
17	SP1548197	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout LH, cpl. with holder		1
18	SP1548200	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout LH, cpl. with holder		1
19	SP1422952	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout LH, cpl. with holder		1
20	SP1584662	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout LH, cpl. with holder		1
21	SP1535009	Tray table, 540mm, Cutout LH		1
22	SP1534974	Tray table, 620mm, Cutout LH		1

Trays

Pos.	Part number	Description	Valid from → until	Qty
23	SP1534975	Tray table, 660mm, Cutout LH		1
24	SP1586047	Tray table, 760mm, Cutout LH		1
25	SPZ177011	Tray Support Fixing Kit cpl.SW 380/505 mm		1
26	Z177025	Fastening-Kit Tray		1
27	SP1590090	Table Bumper high version		1

Hour Counter


1567259.iso

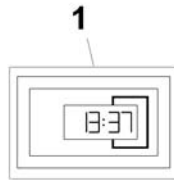

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530275	Hour counter		1

KLICKfix® bag


1562500 iso

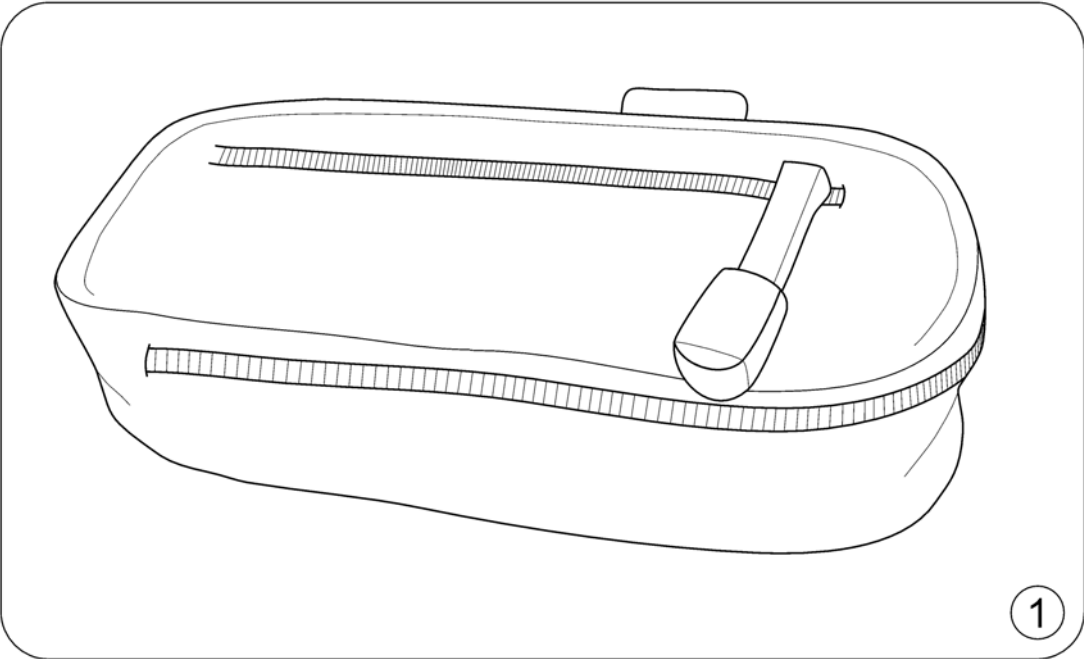

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557532	KLICKfix® bag		1

KLICKfix® bracket for T-Armrest


1571871 iso

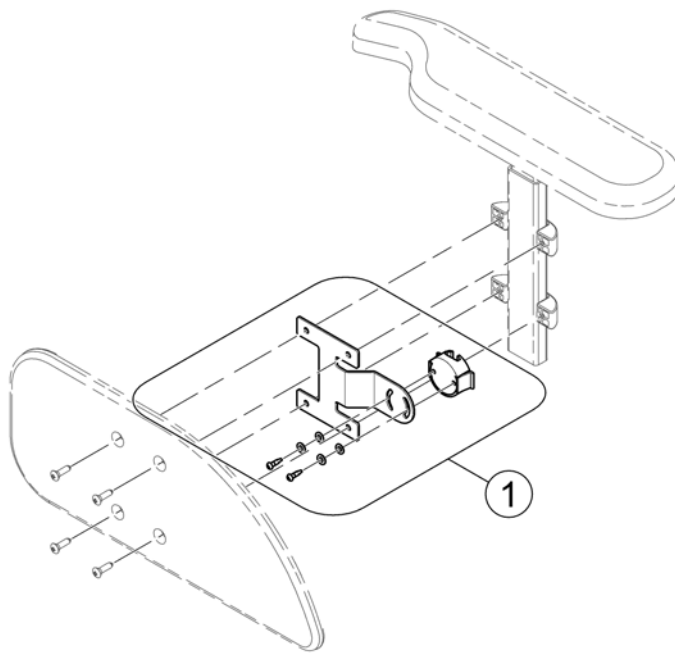

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571678	KLICKfix® bracket		1

KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite)


1586167 Rev

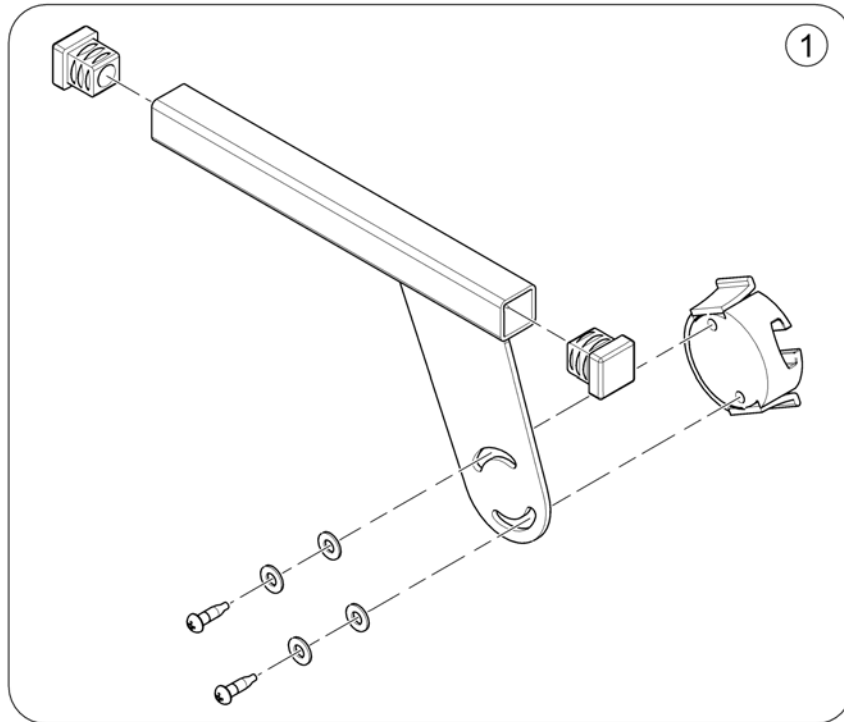

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584497	KlickFix Bracket		1

Toolkit


1567260.110

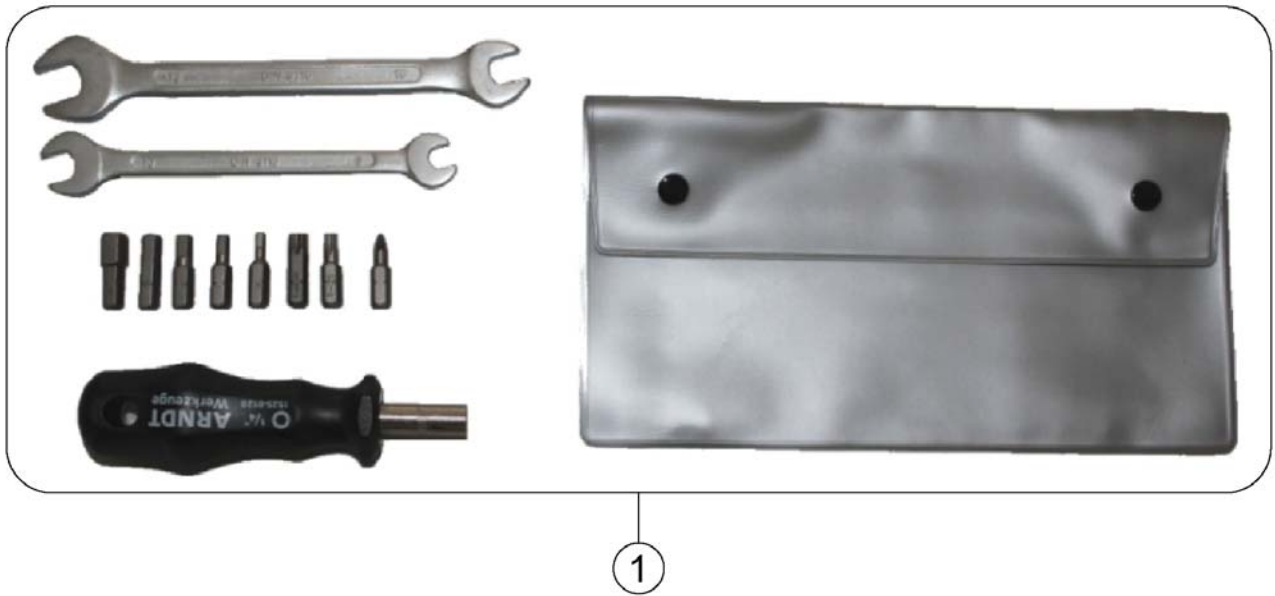

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422868	Toolkit with bag		1

T-Nuts for Accessories Modulite Seat

157-4067 Rev

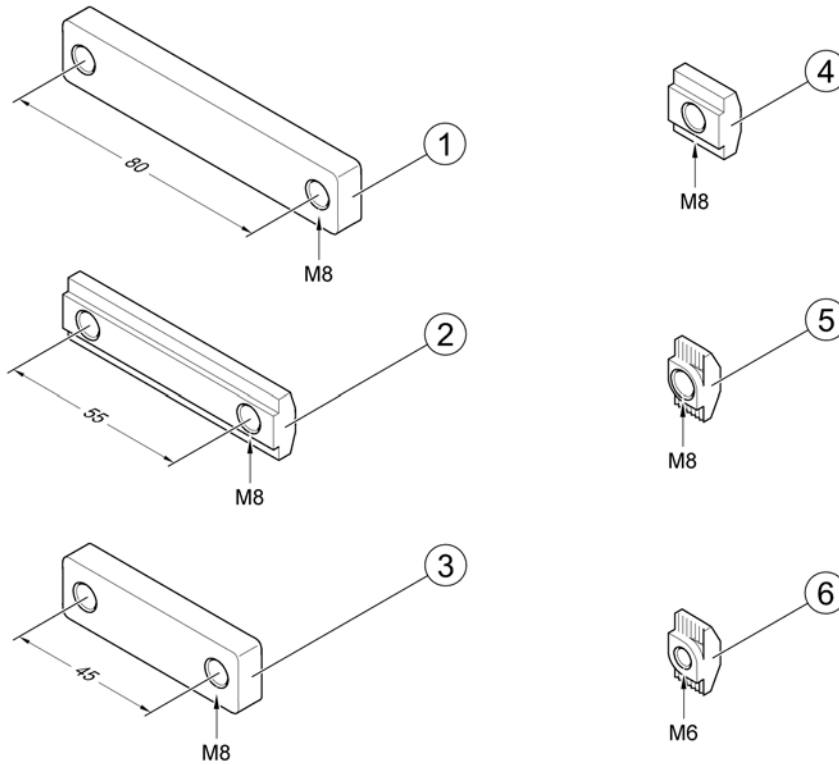

Pos.	Part number	Description	Valid from → until	Qty
1	SP1556401	T-Nut, hole distance 80mm, 2x M8		1
2	SP1572444	T-Nut, hole distance 55mm, 2x M8		1
3	SP1555168	T-Nut, hole distance 45mm, 2x M8		1
4	SP1557341	T-Nut, 1x M8		1
5	SP1573158	T-Nut, 1x M8		1
6	SP1573721	T-Nut, 1x M6		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk