Invacare[®] Fox

FSN 415688, Kit no. 1605416

en	Drive wheel recall kit	Service Manual	1
de	Antriebsrad-Rückrufkit	Service Handbuch	1
es	Kit de retirada de las ruedas motrices	Manual de servicio	2
fr	Kit de rappel de roue motrice	Manuel de maintenance	2
nl	Terugnaset achterwiel	Servicehandleiding	3
pt	Kit de retirada das rodas motrizes	Manual de assistência	3

I Replacing the drive wheels en

Kit:

- 2 wheels
- 2 self-locking nuts
- 2 washers
- 6 distance washers
- 2 plastic caps

CAUTION!

Risk of crushing to hands and feet by weight of wheelchair

- Pay attention to hands and feet.
- Use proper lifting techniques.

CAUTION!

Risk of injury due to uncontrolled movement of the wheelchair

- Switch the wheelchair's power system off (ON-/OFF key).
- Engage the drive motors.
- Secure the wheelchair against rolling away by placing wedges under the wheels.

Tools:

- 19 mm wrench
- 19 mm socket wrench
- flat screwdriver
- oblong wooden blocks (approx. 14 x 14 x 30 cm)

Removing the wheel

1. Prop up wheelchair on multiple wooden blocks. Use proper lifting techniques.
2. Use flat screwdriver to remove plastic cap (5).
3. Loosen and remove nut (4).
4. Remove washer (3).
5. Remove wheel (1) from shaft.


6. Remove remaining washers (2).
7. Discard all parts.
8. Repeat steps for other wheel.

Refitting the wheel

Use only the new parts delivered with the kit.

1. Put three distance washers on shaft.
2. When installing the wheel:
 - Pay attention to correct direction of rotation.
 - Make sure to mount flat side of rim to inside.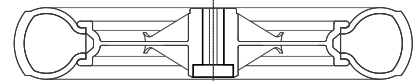

3. Put washer on shaft.
4. Put nut (4) on shaft and tighten to 35 Nm +/-10% Nm. Make sure to fit nut correctly.
5. Put plastic cap on nut.
6. Repeat steps for other wheel.

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DEALER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.

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Invacare AG
Invacare AB
Invacare Limited
Invacare Deutschland GmbH