Rheumatic Hand Control for Invacare® NordBed™

Invacare® NordBed™ Ultra, Invacare® NordBed™ Optimo,
Invacare® NordBed™ Optimo Wide

en	Rheumatic Hand Control for Invacare® NordBed™ User Manual	2
da	Reumatisk håndbetjening til Invacare® NordBed™ Brugsanvisning.....	3
fi	Reumaatikon käsiohjain Invacare® NordBed™ -vuoteelle Käyttöohje	4
it	Pulsantiera reumatica per Invacare® NordBed™ Manuale d'uso	5
nl	Handbediening voor reumapatiënten voor Invacare® NordBed™ Gebruiksaanwijzing	7
no	Revmatisk håndkontroll for Invacare® NordBed™ Bruksanvisning.....	8
sv	Manöverdosa för reumatiker, Invacare® NordBed™ Bruksanvisning.....	10

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved for future reference.

Yes, you can.®

1 Introduction

This manual belongs to an Invacare accessory and is an addition to the manuals of the related Invacare product.

To ensure safety when using this accessory, read the user manual of the related product carefully and follow the safety instructions.

For further information see the user manual of the related product.

2 General Information

The rheumatic hand control can be used alone or in combination with another hand control. An additional hand control is recommended for more advanced functions, like locking or unlocking functions and resetting the system.

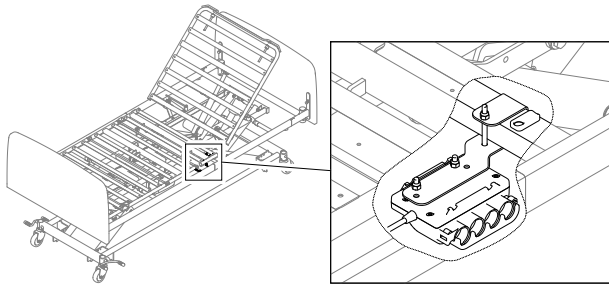
The system consists of:

- 1x Rheumatic hand control
- 1x Junction box
- 1x Adapter cable

3 Installing the Rheumatic Hand Control


Mounting the junction box







-  1x 4 mm Allen key, 1x 8 mm Wrench


1. Place the mounting bracket of the junction box over the square tube of the mattress support platform.
2. Fix the junction box with the bolt and nut.

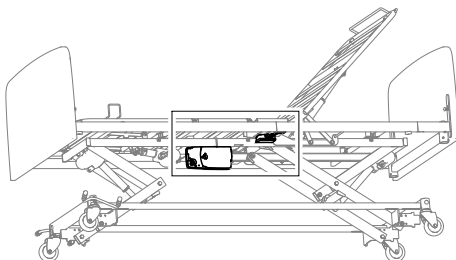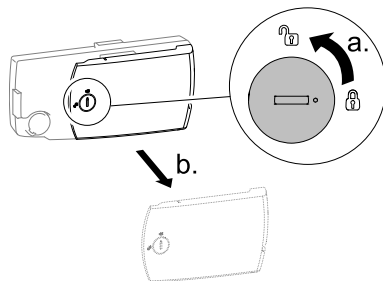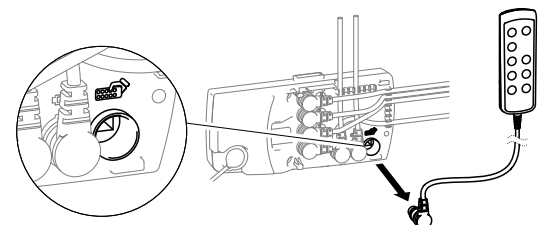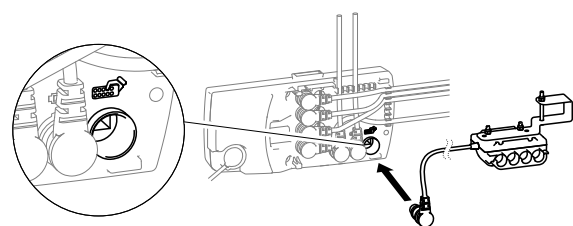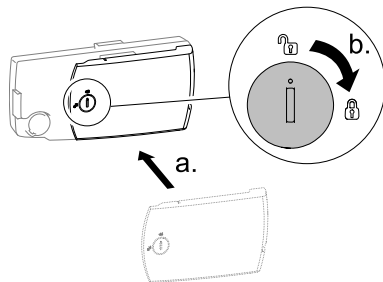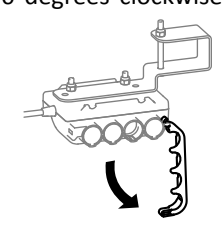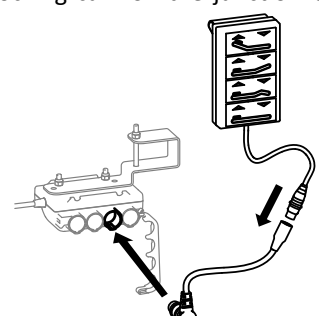Wiring


1.  On the control unit turn the lock 90 degrees counter-clockwise to open the lid and remove it.
2.  Disconnect the cable of the tethered hand control from the control unit.
3.  Connect the cable from the junction box to the control unit in the socket labeled with hand control symbol.
4.  Put the lid back on the control unit and close it by turning the lock 90 degrees clockwise.
5.  Remove the locking cam on the junction box.
6.  With the adapter cable, connect the rheumatic hand control to the yellow marked socket on the junction box.

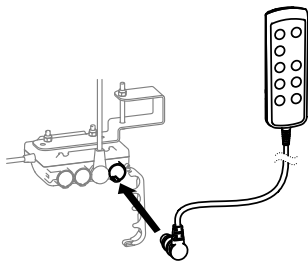7. (optional)

Connect the standard hand control to the empty socket on the junction box.

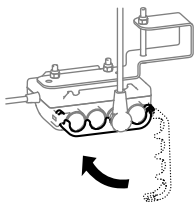8.

Reinstall the locking cam and ensure it is locked in place.

4 Using the Rheumatic Hand Control

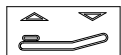
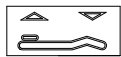
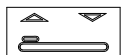
The hand control is equipped with four buttons to operate the basic electrical functions of the bed.

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©2020 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB
Invacare Limited
Invacare Rea AB