Rea[®] Azalea Base

- EN** “Tie down labels”
Instruction sheet
- FR** “Étiquettes de fixation”
Fiche d’instruction
- DE** „Anhängeschilder“
Merkblatt
- IT** “Etichette per l’ancoraggio”
Foglio di istruzioni
- ES** “Fijar etiquetas”
Hoja de instrucciones
- PT** “Etiquetas de refreio”
Folheto de instruções
- SE** “Fastspänningsetiketter”
Instruktionsblad
- NO** “Festemerking”
Anvisningsark
- DA** “Fastsurringsmærkater”
Instruktionsblad

Fig. 1

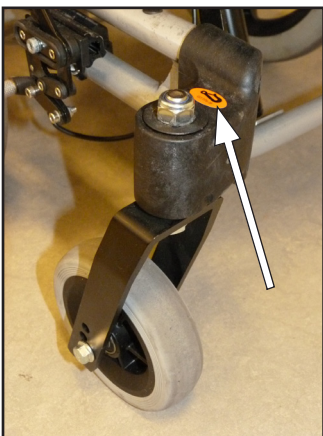
Fig. 2

Fig. 3

Fig. 4

This manual must be given to the user of the product. Before using the product, read this manual and save it for future reference!

Yes, you can.[®]

The Rea® Azalea Base is a part of the Azalea family as a wheelchair base for different seating systems such as individually adapted anatomic seats manufactured by different companies.

The Azalea range is CE-marked, but since the Rea® Azalea Base is not to be considered as a complete product and it is only when the base and the seating system have been combined that an evaluation of the safety can be done, it rests upon the company that mounts the seating system to perform a final risk assessment.

However, if there is an agreement between Invacare and the manufacturer of the seating system that regulates the responsibility between the companies, the product can maintain its CE-marking. In other cases, the company mounting the seating system will be responsible for the CE-marking or that the chair will follow the rules for "Custom-made devices" according to the European Directive 93/42/EEC concerning medical devices.

The Rea® Azalea Base has been crash tested together with the Invacare "Flex 2" standard backrest, "Flo-shape" seat cushion and "MatrX Personal Back". All together with REA neckrests. Invacare can in no way predict the effect of an accident with other configurations.

If the above demands are fulfilled and the judgement is that the product is considered to be crash safe and are able to be transported in a vehicle, the received labels can be attached on the product, as seen on the figures 1-4 on the front side. The labels show where/how the attachment straps should be placed on the chair when it is transported in a vehicle.

For more important information concerning transportation in vehicles, please see the user manual for Azalea. If the user manual is not available, it can be obtained from your Invacare representative or be downloaded from an Invacare website.

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Algol-Trehab Oy
Invacare Rea AB