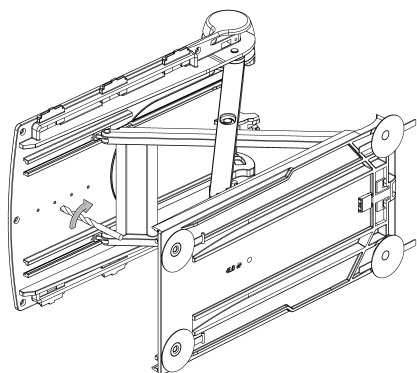
## Aquatec® BELUGA / ORCA

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Before using the product, please read these operating instructions carefully. Pay attention to all instructions, especially those regarding safety, and follow them at all times.

Pay attention also to the operating instructions of the tools used for assembly. Be cautious while drilling holes in the seat plate to not hurt your hands/ fingers.

## SCOPE OF DELIVERY

- Pommel/ Abductionwedge  
Art.-Nr. 1525932: Ø 7 cm  
Art.-Nr. 1525933: Ø 12 cm
- 2. Assembly instructions

*For devices without assembly hole:*

## TOOLS

- Electric drill, Ø 11 mm
- Reamer 15x90° for deflashing

## PREPARATION

1. Remove the backrest and the seat cover mat according to the instructions in the user manual
2. Put the product on the floor or table left side up. To open the scissors press the red button on the spindle-foot support.

**Caution: Risk of pinching with open scissors**

## MONTAGE

1. Drill through the seat plate at the right position for the abduction wedge according to figure 1.
2. Deflash the hole from the top and bottom of the seat plate.
3. Attach the seat plate cover mat.

*For all devices:*

4. Use a pair of scissors to cut a hole in the seat cover mat at the position of the hole in the seat plate.
5. Assemble the backrest.
6. Mount the pommel on the bathlift by putting the pin in the hole in the seat plate. Hold the wedge with one hand and turn the knob clockwise to tighten and lock the wedge.
7. To unmount, turn the knob counter-clockwise. Turn the knob until reaching the upper Stop and pull the abductionwedge upwards out of the hole.

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