

Selecting the correct Invacare sling size



Knowledge bank




Yes, you can.®

Selecting the correct Invacare sling size

Selecting the right size

To select the correct sling size, the most important measurements are the hip width and back height in the sitting position. For slings with belts, the waist circumference is important, too.

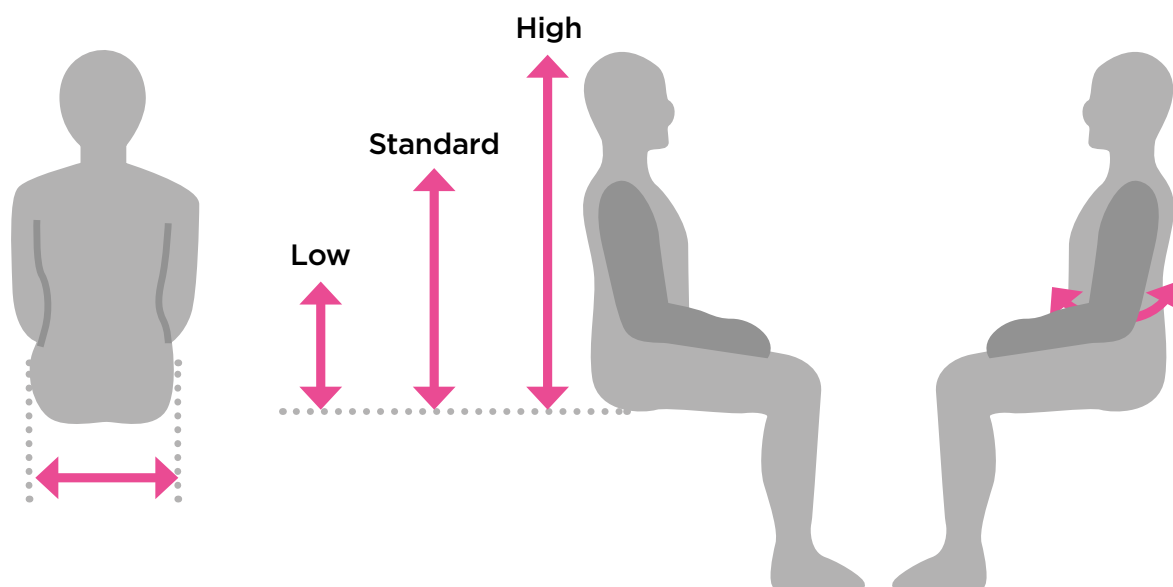

Invacare Slings sizes and safe working load

► Size	XS	S	M	L	XL	XXL
► Safe Working Load	200 kg	200 kg	200 kg	200 kg	250 kg	300 kg

The client's weight needs to be considered mainly for the sling's maximum safe working load. It's inferior to indicate the right sling size.

If no head support is needed, the correct back height is from the lower back until the shoulder/neck. With the need of head support the measurement needs to include the head, too.

► If the sling is too big, the client may slip out. If the sling is too small, the client may still slip out, particularly with less body control. It could also exasperate the client's medical condition. When selecting a size, if your client is in-between sizes, having a slightly smaller sling, could keep your client more secure.

Some medical conditions such as stroke, orthopedic conditions, amputations or certain wounds may affect sling choice.









Selecting the correct Invacare sling size

Measurements

One of the most important factors to perform a safe transfer is the match between hip/trochanter width and the measurement of the sling's opening between the leg support. If between sizes the smaller size may keep the client more secure.

Invacare Slings with divided legs


Universal family with regular opening		XS	S	M	L	XL	XXL
Universal Low	A	820	850	920	950	970	
	B	700	800	935	1075	1140	
	C	280	350	375	405	405	
	D	365	410	480	575	730	
Universal Standard	A	965	1000	1085	1180	1240	
	B	800	915	1050	1095	1240	
	C	405	510	560	585	590	
	D	365	410	480	555	720	
Universal High	A	1040	1080	1425	1555	1600	1650
	B	740	845	1000	1120	1190	1285
	C	615	770	900	1000	1000	1000
	D	310	350	375	450	570	600
Universal High Plus	A	1335	1385	1510	1580	1590	
	B	500	570	660	740	850	
	C	705	880	935	970	970	
	D	305	345	455	565	630	
Small opening		XS	S	M	L	XL	
Amputee	A	1060	1100	1160	1360	1510	
	B	815	930	970	1230	1360	
	C	390	490	520	570	600	
	D	285	320	355	410	500	
Easy-Fit	A	1105	1130	1200	1270	1360	
	B	710	760	800	860	950	
	C	590	630	675	720	760	
	D	150	185	235	245	255	
	E	170	180	190	210	210	
	F	120	150	195	215	225	
Wide opening		XS	S	M	L	XL	
Dress Toileting Low	A	840	870	950	1020	1100	
	B	620	710	800	900	1080	
	C	160	200	230	230	270	
	D	380	430	495	600	760	
Dress Toileting High	E	400	400	465	465	465	
Same dimensions as Dress Toileting Low plus "E"							

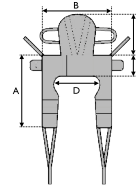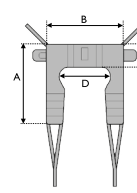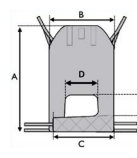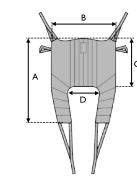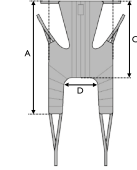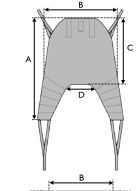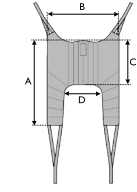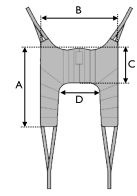

Selecting the correct Invacare sling size


Invacare Slings with undivided legs for full body support

Comfort Family		XS	S	M	L	XL	XXL
Comfort Standard & Comfort in Situ	A	870	960	1020	1160	1170	1200
	B	970	1050	1170	1290	1330	1395
	C	505	550	610	675	745	820
	D	940	1000	1050	1110	1275	1340
Comfort High	A	1040	1150	1395	1445	1460	
	B	970	1050	1170	1290	1330	
	C	510	550	610	675	745	
	D	960	1020	1160	1280	1470	

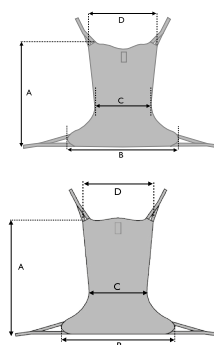


Vest for Standing and Walking Training

		XS	S	M	L	XL
Standing Transfer Vest	A	150	150	180	180	180
	B	750	850	950	1050	1200
Groin Band	C	-	850	-	950	-
	D	-	850	-	350	-

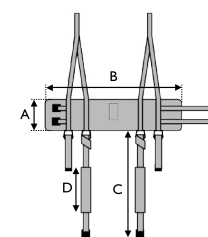

Transfer Support


		S	L
Light Transfer Support	A	930	1120
	B	120	120

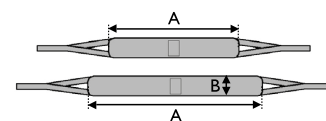

Invacare Slings for standing and raising lifters

		XS	S	M	L	XL
Stand Assist	A	260	260	330	360	360
	B	840	840	895	950	1020
	C	900	950	1170	1270	1810
Transfer Stand Assist	D	-	965	1060	1160	1260

Same dimensions as Stand Assist plus "D"

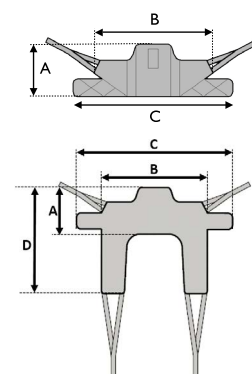

► Size	XS	S	M	L	XL	XXL
► Safe Working Load (kg)	200 kg	200 kg	200 kg	200 kg	250 kg	300 kg

Invacare Limited
Pencoed Technology Park - Pencoed
Bridgend CF35 5AQ - United Kingdom
Tel: +44 1 656 776 222
Fax: +44 1656 776 220
E-mail: uk@invacare.com
Sales Order E-mail: ordersuk@invacare.com
www.invacare.co.uk

Invacare Ireland Ltd
Unit 5 - Seatown Business Campus
Seatown Road - SWORDS
County Dublin - Ireland
Tel. +353 1 8107084
Fax +353 1 8107085
Email: Ireland@invacare.com
www.invacare.ie

© 2021 Invacare International GmbH
All rights reserved.
OptiSling - UK - 04/2021



Yes, you can.®