



erivo<sup>®</sup> carbon



# Life: to live.

erivo carbon – the lightweight and foldable electric wheelchair for urban mobility.

**NEW**



[www.alber.de](http://www.alber.de)



# Big city? No big deal.

**Your compact companion for everyday urban life.**

The erivo carbon is made for a life on the move. For busy streets, short and long journeys, spontaneous errands, and all the places that make up your everyday life in the city.

The erivo carbon can be folded up compactly in a single step and set up again just as quickly. This makes it ideal for people who want to move

around the city flexibly and value an aid that feels as modern as it looks. Its clean lines, high-quality workmanship, and smart features immediately show that the erivo carbon is more than just an electric wheelchair. It is an urban companion that fits seamlessly into your everyday life – whether on the train or on the way to your favorite café.

# erivo carbon is the right choice for you if:

- ✓ you are often out and about in the city and want a particularly lightweight, foldable wheelchair that is easy to take with you.
- ✓ you still have some walking ability but need flexible support in an urban environment, for example when shopping or traveling longer distances in the city.
- ✓ you want to continue to lead an active and independent lifestyle despite limited walking ability.
- ✓ you suffer from irregular, brief episodes of loss of balance but want to be able to cover everyday distances safely.
- ✓ You need a compact electric wheelchair temporarily and/or situationally after an accident or neurological illness.
- ✓ You are looking for an electric wheelchair that is just as impressive indoors as it is on your journeys through the city.



# urban



## STRONG DESIGN

### Arrive in style.

The erivo carbon not only looks modern, it feels that way too. Its clean lines, slim shape, and high-quality paint finish make it a companion that blends effortlessly into any urban environment.

With its contemporary design language, it sets new standards in the field of mobility aids and shows that function and style are not mutually exclusive, but can work together perfectly.

- ✓ **Two-tone paint finish**  
in amber metallic and carbon black
- ✓ **Modern and elegant frame design**



# & agile



## LIGHT

### Light in weight. Heavy on impact.

The frame, made entirely of carbon, makes the erivo carbon both ultra-light and dimensionally stable. With a total weight of only about 19.9 kg (including battery and bag), it is easy to handle, lift, or stow in a car.

This lightness opens up new flexibility in everyday life. Ideal for people who need an aid that gives them real independence rather than extra weight.

- ✓ **Carbon frame**  
lightweight and stable
- ✓ **High-quality materials**

## DYNAMIC

### Feel the ride. Feel the finesse.

The erivo carbon responds precisely to every movement and can be controlled intuitively, so every ride feels safe and confident.

Whether it's narrow passages, changes of direction, or short stops in everyday city life – everything flows smoothly and effortlessly.

The finely tuned controls enable smooth starting, steady acceleration, and controlled braking. The erivo carbon also reliably masters inclines of up to 16 percent, which are common in urban areas.

- ✓ **Joystick control**  
intuitive and precise steering
- ✓ **Gradients of up to 16%**



# clever &

CLEVER

## One button. Great flexibility.

The erivo carbon can be folded up compactly in a single step and set up again just as easily. As a foldable electric wheelchair, it is ideal for transport in cars, buses, and trains, or for space-saving storage at home. The armrests and footrest can be easily folded up and down, making getting in and out particularly comfortable. Perfect for moving between your home, the city, and public transport.

- ✓ **Height-adjustable footrest** and foldable armrest
- ✓ **Folds with just one hand**

## Push it. You'll love it.

If you want, the erivo carbon can be converted into a rollator. The motors can be easily disengaged so that you can push and maneuver it easily, with almost no resistance.

This gives you additional freedom in everyday life, when walking short distances around your home, in a café, or anywhere else where you prefer to push rather than ride.

- ✓ **Disengage motors** manually or with a key combination
- ✓ **Easy to push**



# flexible

## All inclusive. Even free hands.

The erivo carbon offers practical storage space for shopping and personal items. The basket under the seat and the magnetically attached bag on the backrest ensure that everything is safely stowed away and your hands remain free.

Practical compartments keep things tidy, while the included carrying strap makes it easy to transport the wheelchair when not in use.

- ✓ **Bag with magnetic closure**  
10 l capacity, load capacity up to 10 kg
- ✓ **Basket under the seat**





# Where there's a coffee shop ...

The erivo carbon accompanies you on the very paths that make up your city life. To the bakery on the corner, your favorite café, the weekly market, or a quick shopping trip two streets away. Precise steering makes navigation easy, even when things get tight, busy, or hectic. With a range of up to 25 kilometers, you stay flexible all day long, even if your journeys take longer than planned.

The erivo carbon also remains pleasantly flexible indoors. Thanks to the rollator function, which can be activated at the touch of a button, it is easy to push and maneuver precisely. Ideal for narrow aisles, small shops, or quick maneuvers at home.



For the city



For indoor use



# ... there's a way.

With its compact dimensions, it fits into small shops and on narrow sidewalks – anywhere urban life takes place. And when you switch between bus, train, or car, it folds up in a single motion and sets up just as quickly.

The erivo carbon makes urban mobility easy, flexible, and second nature. Just like your lifestyle.



”

With the erivo carbon, I am completely flexible in the city again. Lightweight, quick to fold up, and just as modern as I want it to be.

**Ute B. from Plön**

What I love most about the erivo carbon is how easy and uncomplicated everything is. Just fold it up, take it with you, and continue on your way. Especially in the city, between shopping, cafés, and short trips in between, it gives me a really good feeling of freedom and flexibility.

**Sabine L. from Konstanz**

“







# Impressive on the outside, stunning on the inside.

The erivo carbon impresses not only with its design, but above all with what's inside it. Every detail is designed to make everyday life easier, safer, and more flexible.

From the intelligent folding mechanism to the precise controls and well-thought-out comfort features, it's clear that this isn't just a matter of design, but of thoughtful engineering.

SCOPE OF DELIVERY AND COMPONENTS

# The perfect frame for performance.

### Foldable armrests

For comfortable getting in and out.



### Height-adjustable and foldable footrest

For an ergonomically ideal sitting position.



### Safety belt

For stable support and a secure seating position.

### Bag on the backrest

With magnetic lock and additional storage space.



### Battery with integrated rear light

Range up to 25 km (20 Ah / 480 Wh)  
Alternatively, lightweight battery with 15 km range (12 Ah / 288 Wh).



### Charger

Compact charger for charging the battery or wheelchair.

### Light and USB-A/USB-C connection

For visibility and smartphone power supply.



### Carbon frame

Both sturdy and lightweight:  
The entire frame is made of high-quality carbon.



### Basket under the seat

Practical storage space directly accessible from the sitting position.

\* In accordance with ISO 7176, actual range depends on the surface, speed, weight of the user, temperature and terrain.

CONTROLS

# Let's get rolling:

The erivo carbon's controls are clearly laid out and intuitive to use. The joystick allows you to control the speed and direction of travel smoothly and precisely, even on narrow paths, when changing direction or in busy city traffic.

The precise calibration ensures a safe riding experience and you remain in control at all times.



Joystick:  
intuitive control

Digital display:  
built into the armrest

FUNCTIONS



**Speed selection**  
Increase or decrease speed and hold speed up for three seconds to activate cruise control.



**ON/OFF button**  
Press and hold for three seconds to turn on or off.



**SOS button**  
Press and hold for three seconds to sound an ALARM SIGNAL.



**Activate the rollator function:**  
Press and hold both arrow buttons simultaneously for three seconds.



**FUNCTION button**  
Switch between NORMAL MODE <-> NOVICE MODE (Speed preset limited to 1 or 2).



**HORN button**  
An audible warning tone sounds.



You can also find both battery options in the „Accessories” section

## Technical Data

erivo carbon R20 / 12Ah battery

erivo carbon R20 / 20Ah battery

### Totalweight

Weight (includes battery and case) 19,9 kg

20,8 kg

Max. user weight 150 kg

150 kg

### Range\*

15 km

25 km

### Battery

12 Ah, 288 Wh



20 Ah, 480 Wh

Battery weight

2,5 kg

3,4 kg

Charger

3 A Charger / IPXO

3 A Charger / IPXO

Charging time

ca. 4,5 h (installed or removed)

ca. 7,5 h (installed or removed)

### Dimensions

Dimensions when ready to ride

63 x 97 x 100 cm (W x L x H)

Dimensions when folded

63 x 72 x 45 cm (W x L x H)

Seat

44 x 43 x 5 cm (W x L x H)



\* In accordance with ISO 7176, actual range depends on the surface, speed, weight of the user, temperature and terrain.



# Accessories

## 01. Cup holder

The cup holder with an additional compartment ensures that everything you need stays within easy reach.

## 02. Smartphone holder

The smartphone mount securely holds your phone within your field of vision. Perfect for navigation, calls, or music while driving.

## 03. 12 Ah replacement battery\*

The 12 Ah (288 Wh) replacement battery is IATA-compliant and can be taken on board an airplane.

## 04. 20 Ah replacement battery

The 20 Ah (480 Wh) replacement battery takes you up to 25 km.

## 05. Remote Control



With the remote control, the erivo carbon can also be operated by an accompanying person or from the couch. Ideal for additional support in everyday life or on longer trips.

## 06. Heated seat cushion

The heated seat cushion provides pleasant warmth in cooler temperatures and noticeably increases seating comfort.

## 07. Additional front light

The additional front light improves visibility ahead and ensures even greater safety during twilight or darkness.

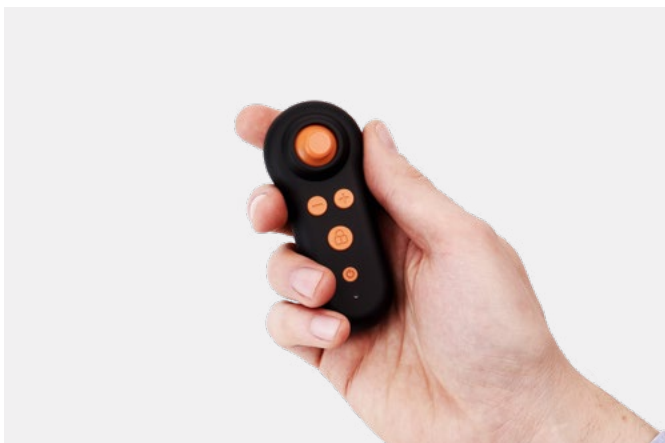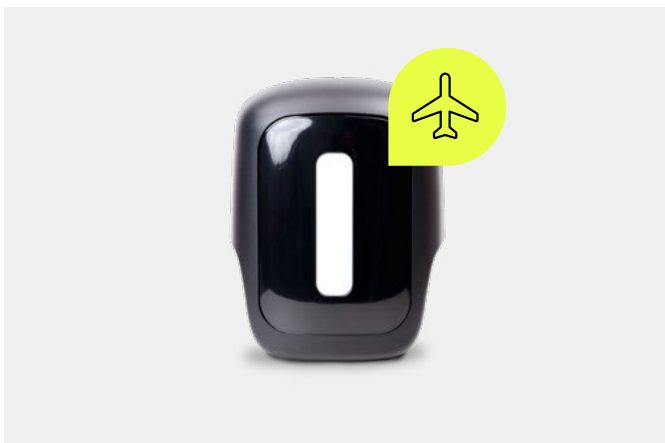

\* In accordance with ISO 7176, actual range depends on the surface, speed, weight of the user, temperature and terrain.



# Powered by alber.

## Simply mobile.

As the market leader in mobility aids for wheelchairs, Alber GmbH has more than 35 years of experience in the electromobility. We are a trusted partner to health insurance companies and collaborate with healthcare suppliers worldwide. From the initial consultation to ongoing service: You can rely on us.

Visit our website for more information about our products and to find answers to all your questions and to locate Alber partners worldwide in your country.

The team at Alber GmbH looks forward to meeting you and is happy to provide you advice and assistance.

[www.alber.de](http://www.alber.de)

# Not a perfect fit?

It is entirely possible, of course, that another Alber product is simply a better fit for you and your individual requirements. We'll be happy to help you find it. Do you perhaps already have a customised wheelchair and want to continue using it? Discover other mobility solutions from Alber at [www.alber.de](http://www.alber.de).



Ongoing support:

## e-fix®

The versatile add-on drive for your individual wheelchair.



You are here:

## erivo® carbon

The lightweight and foldable electric wheelchair for urban mobility.



Your outdoor companion:

## erivo®

The comfortable and foldable electric wheelchair for new horizons.



## **erivo carbon – the lightweight and foldable electric wheelchair for urban mobility.**

The erivo carries the CE mark for conformity with the European Medical Devices Regulation (MDR) 2017/745.

Our quality commitment – your safety! Alber is certified under the current quality management standard ISO 13485 for medical devices. Our environmental management system, which is certified under ISO 14001, ensures our products are manufactured in an environmentally responsible manner.

**Alber GmbH**  
Vor dem Weißen Stein 14  
72461 Albstadt-Tailfingen, Germany  
info@alber.de