

Spare Parts Catalogue

Version: PartList_EFIXE35_20190624a NO





Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.




Yes, you can.[®]

TABLE OF CONTENTS


(Drive wheels E35/E36)	5
(Drive wheels E35/E36 spare parts)	6
(Control unit e-fix E35/E36)	7
(Control unit e-fix E35/E36 spare parts)	8
(Battery fixation with interface and charger)	9
(Battery fixation with interface and charger Ersatzteile)	10
(Charger spare parts)	11
(Battery pack E35/E36)	12
(Battery pack E35/E36 spare parts)	13
(Accessories)	14
(Anti-tippers with jack-up function (Pair) spare parts)	15
(anti-tippers without jack-up function spare parts)	16
(Spoke cover E36 24" with Alber decor complete spare parts)	17
(Anti-tippers without jack-up function black 12mm (pair) spare parts)	18
(Anti-tippers with jack-up function (Pair) black spare parts)	19
(Swivel arm for control unite e-fix E35/E36 spare parts)	20

(Drive wheels E35/E36)

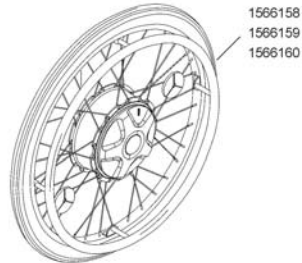

-	(Drive wheels E35/E36 spare parts)	1
1566158	DRIVHJUL 24 E-FIX E35KPL	1
1566159	DRIVHJULL 22 RESERVEDELE-FIX E35	1
1566160	DRIVHJUL 24 KPL e-fixE36	1

(Drive wheels E35/E36 spare parts)

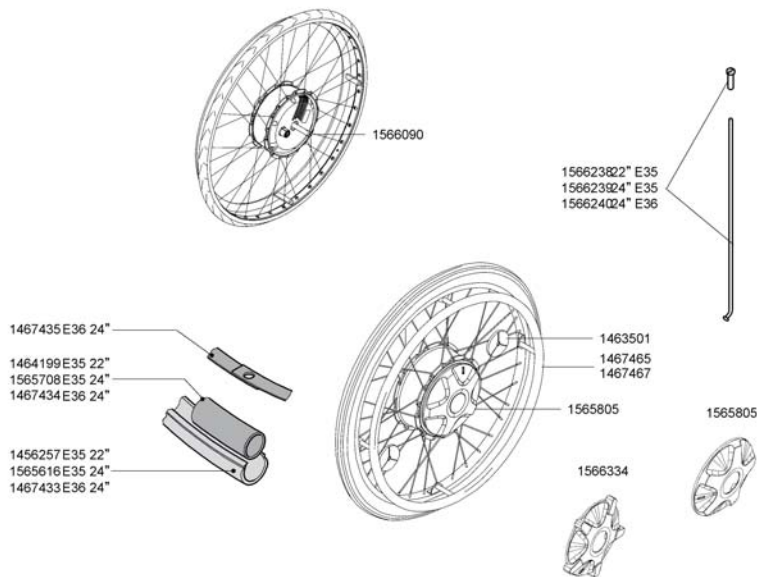


1565708	INNLEGG 24"x1.3/8" PKTSIKKER E35	1
1463501	REFLEKS EF EM HJUL	1
1464199	INNLEGG PKT FRI 22'INDRE DEL E-FIX	1
1467433	DEKK 24" MARATHON PLUSe-fix 26	1
1467434	SLANGE MED BIL VENTILvalve 38/47-540 E26	1
1467435	FELGBÄND 24 SchwalbeE-FIX	1
1467464	(Rim Tour silver 22" (489-21))	1
1467465	DRIVRING 24 E25 E26E-FIX	1
1467467	DRIVRING 22 E25 E-FIX	1
1565107	(Tyre C63 grey 22 x 1 3/8 (37-489))	1
1565616	DEKK 24X1X1 3/8GRÅ/SORT E-FIX	1
1565805	DEKSEL FRIKOBLING HVITE35	1
1566090	(Stop screw stub axle e-fix)	1
1566117	(Rim Tour silver 24", eyed)	1
1566118	(Rim Tour black 24", reinforced and eyed)	1
1566238	(Spoke set E35 22" Leader 5 spokes + nipples)	1
1566239	EIKESETT KPL 5 STKE35 24	1
1566240	(Spoke set E36 24" Race 5 spokes + nipples)	1
1566334	(Coupling ring E35/E36 with extension complete)	1




-	(Control unit e-fix E35/E36 spare parts)	1
1566150	STYREBOKS E35/E36 KPL	1

(Control unit e-fix E35/E36 spare parts)

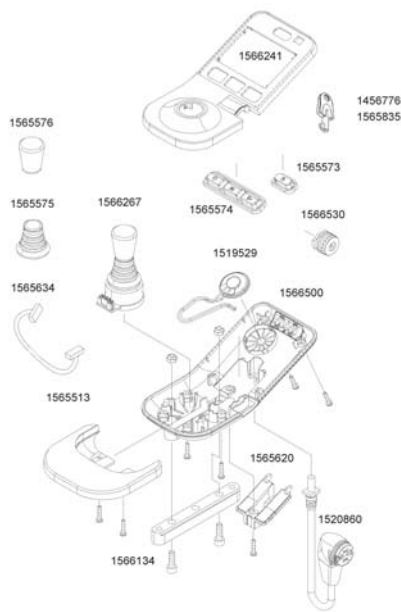


1456567	(Screw Ejot PT K 30 x 8 WN 1451)	1
1456776	LÅSENØKKEL E35 E36	1
1459095	(Screw for width adjustment X6)	1
1459626	(Hexagon nut ISO 4032 - M5 - 6 AU)	1
1463520	(Screw Ejot PT K 30 x 14 WN)	1
1463627	(Screw Ejot PT K 30 x 12 WN 1423)	1
1520860	SPIRALKABEL VIAMOBIL V25VIAMOBIL V25	1
1565513	(Hand rest for control unit E35/E36)	1
1565514	(Headlight glass for control unit E35/E36)	1
1565573	SWITCH AV/PÅ STYREBOKSE35/36	1
1565574	SWITCH MENY STYREBOKSE35/36	1
1565575	GAMASJE JOYSTICK E35/E36E-FIX E35	1
1565576	JOYSTICK KJEGLE StandardE35/E36	1
1565620	(Control unit service window E35/E36)	1
1565634	(Cable for joystick E35/E36)	1
1566134	(Rail for control unit E35/E36)	1
1566241	DEKSEL TOP TIL STYREBOKSE35/E36	1
1566267	(Joystick E35/E36 complete)	1
1566500	DEKSEL UNDERSIDE TILSTYREBOKS E35/36	1

(Battery fixation with interface and charger)

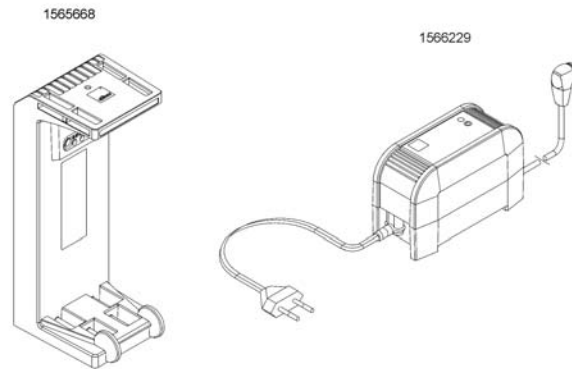


-	(Battery fixation with interface and charger Ersatzteile)	1
1565668	BATTERIHOLDER M KABLERE-FIX E35/E36	1
-	(Charger spare parts)	1
1566229	LADER E-FIX E35E-FIX E35	1

(Battery fixation with interface and charger Ersatzteile)

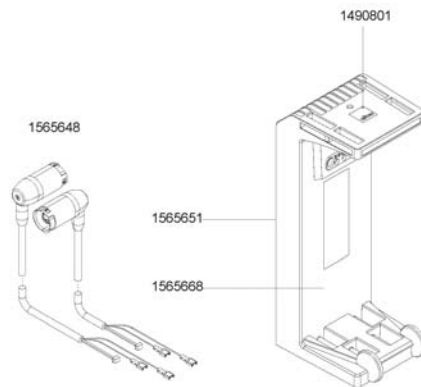


1490801	ALBER LOGO 17 x 17mmE-FIX E35	1
1565651	DEKSEL V BATTERIE-FIX E35/E36	1
1565668	BATTERIHOLDER M KABLERE-FIX E35/E36	1
1565648	MOTORKABEL E-FIX E35	1
1566258	VELCRO BÄND KIT SB40-50E-FIX E35	1
1566313	VELCROBÄND KIT < SB40EFIX E35	1
1566314	VELCROBÄND KIT SB50+E-FIX E35	1

(Charger spare parts)

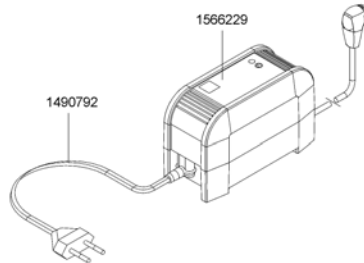


1566229	LADER E-FIX E35E-FIX E35	1
1489256	(Mains cable charger E25/E26 USA/CN)	1
1490792	(Main cable for charger, EU)	1
1490793	(Mains cable charger USA e-motion M15/PS3025)	1
1490794	(Mains cable for charger e-motion M15 PS3025)	1
1490795	(Mains cable for charger e-motion M15 PS3025 AU)	1
1520179	(Mains cable for charger 1.0A USA/Japan)	1
1520180	(Mains cable for charger 1.0A UK-Version)	1
1520181	(Mains cable for charger 1.0A AU)	1
1520182	NETTKABEL LADER V15 V25E35 EU	1
1592355	(Charger e-fix E35/E36/V25 incl. AU mains cable)	1
1592356	(Charger e-fix E35/E36/V25 incl. UK mains cable)	1

(Battery pack E35/E36)

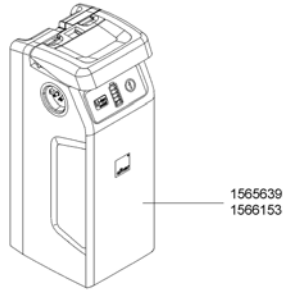


-	(Battery pack E35/E36 spare parts)	1
1565639	BATTERIPAKKE E35/E36,6Ah	1
1566153	BATTERIPAKKE E35/E36,7,5Ah	1

(Battery pack E35/E36 spare parts)

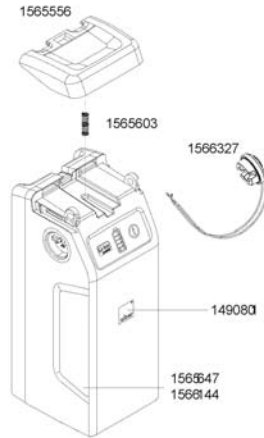


1490801	ALBER LOGO 17 x 17mmE-FIX E35	1
1565556	HÅNDTAK BATTERIBOKS E35/E36	1
1565603	FJÆR BATTERIPAKKEE35/E36	1
1565647	(Sticker on handle standard battery-pack 6,0 Ah)	1
1566144	(Sticker handle powerline battery-pack 7,5 Ah)	1
1566327	KAPPE MED KABEL FORADAPTERKABEL E35/36	1

Alber E-Fix E35 /E36 |
(Accessories)

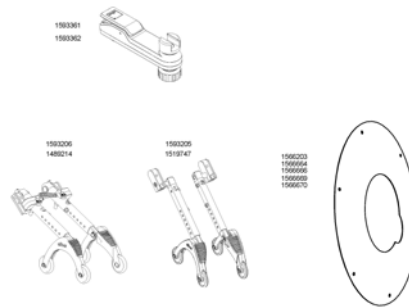


-	(Anti-tippers with jack-up function (Pair) black spare parts)		1
-	(Anti-tippers with jack-up function (Pair) spare parts)	→ 17/06/19	1
1593206	(black anti-tippers with jack up function (pair)) <i>Replaces 1489214</i>	17/06/19 →	1
-	(Spoke cover E36 24" with Alber decor complete spare parts)		1
-	(anti-tippers without jack-up function spare parts)	→ 17/06/19	1
1593361	(Swivel arm for control unite e-fix E35/E36)		1
1566203	EIKEBESKYTTER 22" HJULE-FIX E35		1
-	(Anti-tippers without jack-up function black 12mm (pair) spare parts)		1
1593205	(Anti-tippers without jack-up function black 12mm (pair)) <i>Replaces 1519487</i>	17/06/19 →	1
1566664	EIKEBESKYTTER 24 BLANKE-FIX		1
1593362	(Swivel arm for control unit e-fix E25/E26 (incl. distance set))		1
1566666	EIKEBESKYTTER 24 STKE-FIX E35 MED FESTE		1
1566669	(Spoke cover E36 24" with Alber decor)		1
-	(Swivel arm for control unite e-fix E35/E36 spare parts)		1
1566670	(Spoke cover E36 24" with Alber decor complete)		1

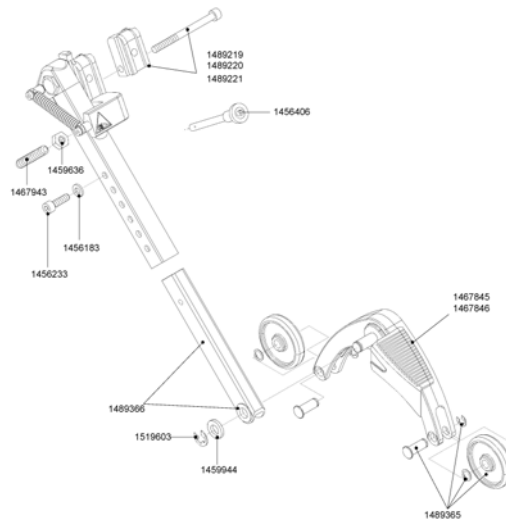(Anti-tippers with jack-up function (Pair) spare parts)


1456183	(Washer DIN 125 - A 6,4)	1
1456233	SKRUE TIPPESIKRING DIN7984 M 6 x 20	1
1456406	LÅSESPLINT ALBERE-FIX E35	1
1459636	(Hexagon nut ISO 4032 - M8 vz)	1
1459944	(Washer DIN 988 - 10 x 16 x 1)	1
1467845	TIPPESIKRING ARM HØ UTENHJUL	1
1467846	TIPPESIKRING ARM VE UTENHJUL	1
1467943	(Threaded pin M 8 x 45 DIN 913 - 45 H)	1
1489214	(Anti-tippers with jack-up function (pair))	1
1489221	ADAPTER TIPPESIKRING JACK UP 12MM BRED	1
1489365	HJUL TIL TIPPESIKRINGM/JACK UP FUNKSJON	1
1489366	(Slide rod anti-tippers mounted)	1
1519603	LÅSERING TIPPESIKRINGDIN 6799 Gr.8 VZ	1

(anti-tippers without jack-up function spare parts)

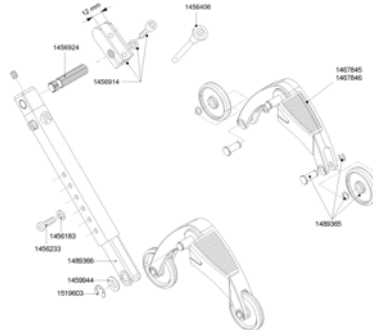

1456183	(Washer DIN 125 - A 6,4)	1
1456233	SKRUE TIPPESIKRING DIN7984 M 6 x 20	1
1456406	LÅSESPLINT ALBERE-FIX E35	1
1456914	FESTESKO 12MM TIPPESIKRALBER	1
1456924	KOBLINGSPINNE ANTI-TIPPEE-MOTION E15/E20	1
1459944	(Washer DIN 988 - 10 x 16 x 1)	1
1467845	TIPPESIKRING ARM HØ UTENHJUL	1
1467846	TIPPESIKRING ARM VE UTENHJUL	1
1489365	HJUL TIL TIPPESIKRINGM/JACK UP FUNKSJON	1
1489366	(Slide rod anti-tippers mounted)	1
1519603	LÅSERING TIPPESIKRINGDIN 6799 Gr.8 VZ	1
1519747	(Anti-tippers without jack-up function 12mm (pair))	1

(Spoke cover E36 24" with Alber decor complete spare parts)

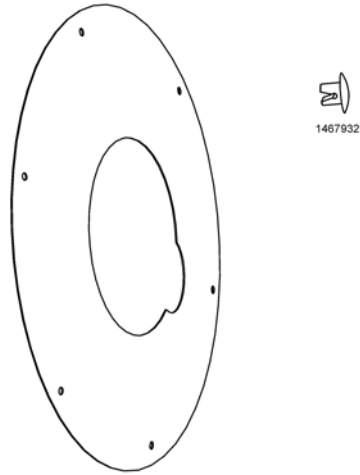


1467932


FESTE FOR EIKEBESKYTTERALBER E-MOTION

1

(Anti-tippers without jack-up function black 12mm (pair) spare parts)

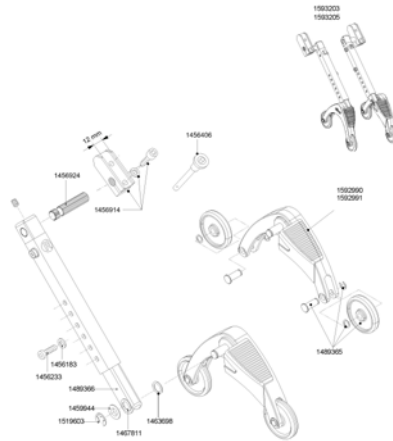


1456183	(Washer DIN 125 - A 6,4)	1
1456233	SKRUE TIPPESIKRING DIN7984 M 6 x 20	1
1456406	LÅSESPLINT ALBERE-FIX E35	1
1456914	FESTESKO 12MM TIPPESIKRALBER	1
1456924	KOBLINGSPINNE ANTI-TIPPEE-MOTION E15/E20	1
1459944	(Washer DIN 988 - 10 x 16 x 1)	1
1463698	(DIN 988 - 10 x 16 x 0,3 - St)	1
1489365	HJUL TIL TIPPESIKRINGM/JACK UP FUNKSJON	1
1489366	(Slide rod anti-tippers mounted)	1
1519603	LÅSERING TIPPESIKRINGDIN 6799 Gr.8 VZ	1
1592990	(Support angle right for anti-tippers (black))	1
1592991	(Support angle left for anti-tippers)	1
1593203	(Anti-tippers without jack-up function black (pair))	1
1593205	(Anti-tippers without jack-up function black 12mm (pair))	1

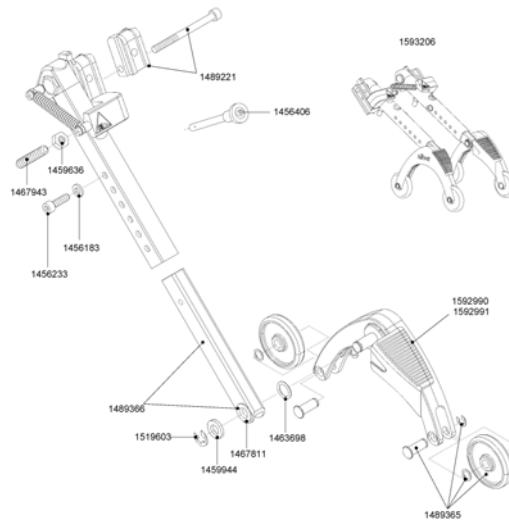(Anti-tippers with jack-up function (Pair) black spare parts)

1456183	(Washer DIN 125 - A 6,4)	1
1456233	SKRUE TIPPESIKRING DIN7984 M 6 x 20	1
1456406	LÅSESPLINT ALBERE-FIX E35	1
1459636	(Hexagon nut ISO 4032 - M8 vz)	1
1459944	(Washer DIN 988 - 10 x 16 x 1)	1
1463698	(DIN 988 - 10 x 16 x 0,3 - St)	1
1467811	(friction bearing Iigus form F GFM-1212-035)	1
1467943	(Threaded pin M 8 x 45 DIN 913 - 45 H)	1
1489221	ADAPTER TIPPESIKRING JACK UP 12MM BRED	1
1489366	(Slide rod anti-tippers mounted)	1
1519603	LÅSERING TIPPESIKRINGDIN 6799 Gr.8 VZ	1
1592990	(Support angle right for anti-tippers (black))	1
1592991	(Support angle left for anti-tippers)	1
1593206	(black anti-tippers with jack up function (pair))	1

(Swivel arm for control unite e-fix E35/E36 spare parts)

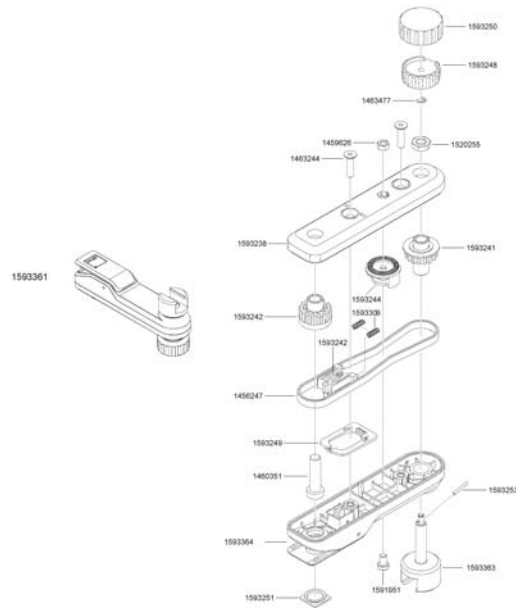

1456247	(Toothed belt for swivel arm)	1
1459626	(Hexagon nut ISO 4032 - M5 - 6 AU)	1
1460351	(DIN 7984 - M 8 x 30 - 8.8 vz)	1
1463244	(DIN 7991 - M 5 x 20 - 8.8 vz)	1
1463477	(Securing disk DIN 6799 - GR. 3,2 galvanized)	1
1520255	(Hexagon nut ISO 4035-M8-04-A2K)	1
1591951	(Screw SHR-DIN7984-08.8-IS3-A2R-M5x8-TUF-FLECK)	1
1593238	(Housing lower part)	1
1593241	(Pulley with sleeve)	1
1593242	(Pulley for latch with sleeve)	1
1593244	(Latch)	1
1593247	(Klemmung für Riemenscheibe)	1
1593248	(Clamping handwheel)	1
1593249	(Plastic spring)	1
1593250	(Handle cover)	1
1593251	(Stopfen Alber Logo)	1
1593252	(Cylindrical pin ISO 2338 3m6x10)	1
1593253	(Grooved dowel pin Würth ISO 8745 2x16)	1
1593309	(Compression spring RDF-1341)	1
1593361	(Swivel arm for control unite e-fix E35/E36)	1
1593362	(Swivel arm for control unit e-fix E25/E26 (incl. distance set))	1
1593363	(Socket for control unite, premounted washer)	1
1593364	(Housing upper part, premounted control knob)	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånå
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk