

Spare Parts Catalogue

Version: 2016/07/28b NO



KÜSCHALL COMPACT_{attract}



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

(INFORMATION)

(INFORMATION)	5
(Abbreviations)	6
(Frame colours)	7

(CHASSIS)

(CHASSIS)	8
(Front frames standard)	9
(Cross brace single, SB36-42)	12
(Cross brace double, SB44-50)	14
(Rear frames)	16
(Rear wheel fixation)	17

(SEAT)

(SEAT)	18
(Seat cover with 1 Velcro band (soft band))	19
(Seat cover with 1 Velcro band (soft band) for seat bag)	21
(Seat cushion standard)	23

(BACKREST)

(BACKREST)	25
(Push handles for backrest standard)	26
(Push handles for backrest angle adjustable)	27
(Push handles for backrest foldable)	28
(Push handles for backrest foldable and angle adjustable)	29
(Push handles height adjustable)	30
(Push handles height adjustable for backrest angle adjustable)	31
(Küschall backrest parts for Matrx backs)	32
(Push handles, inclined)	33
(Backrest stabilization bar)	34
(Back cover, standard)	35
(Bands to adjustable backrest)	36
(Cover to adjustable backrest)	37

(LEGRESTS)

(LEGRESTS)	38
(Legrest hanger)	39

(FOOTREST)

(FOOTREST)	40
(Footrest 2-piece)	41
(Footrest 2-piece, angle adjustable)	43
(Footbands)	44

(SIDERESTS)

(SIDERESTS)	45
(Clothes-guards composite)	46
(Clothes-guards carbon)	47



Yes, you can.®

TABLE OF CONTENTS

(Mudguards composite)	48
(Mudguards carbon)	49
(Mudguards composite removable)	50
(Mudguards carbon removable)	51
(Siderest height adjustable)	52
(Siderest with desk function)	53
(Tubular armrest)	54

(FRONT WHEELS)

(FRONT WHEELS)	55
(Front wheel supporter)	56
(Front wheels)	57

(REAR WHEELS)

(REAR WHEELS)	58
(Active wheel)	59
(24" Kuschall rear wheels complete)	60
(High Performance wheel)	61
(Spinergy LXX wheel)	62
(Drumbrake wheel)	63
(Handrim)	64
(Fixation handrim)	65
(Tyre)	66
(Inner tube and rim protector)	67
(Spoke guard)	68
(Quick release axle and bearing)	69

(BRAKES)


(BRAKES)	70
(Brakes)	71
(Drum brake)	72

(OPTIONS & ACCESSORIES)

(OPTIONS & ACCESSORIES)	73
(Tipper aid)	74
(Antitipper)	75
(Cane holder)	76
(Therapy table)	77
(Seat belt)	78
(Chair accessories)	79
(Fixation: screws, ...)	80
(Socket head cap screws)	81
(Countersunk screws)	84
(Button head screws)	86
(Hex-head screws, diverse, ...)	88
(Washers)	90
(Nuts)	92
(Pins, rivets, ...)	94


1	-	(Abbreviations)	1
2	-	(Frame colours)	1

(Abbreviations)

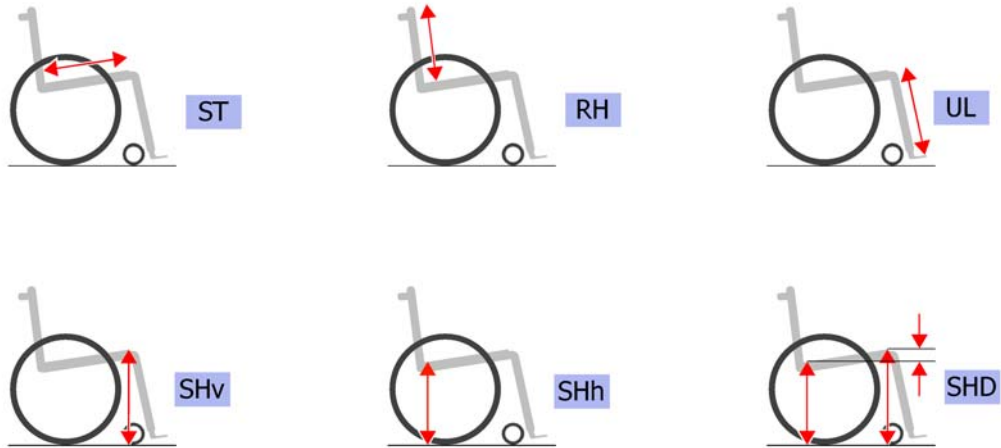


L	-	(Left side (sitting in the wheelchair))	0
R	-	(Right side (sitting in the wheelchair))	0
L+R	-	(Left and right sides (paire))	0
SB	-	(Seat width (2 digits in cm or 3 digits in mm))	0
ST	-	(Seat depth (2 digits in cm or 3 digits in mm))	0
RH	-	(Backrest height (2 digits in cm or 3 digits in mm))	0
UL	-	(Knee-to-heel length (2 digits in cm or 3 digits in mm))	0
SHv	-	(Front seat-to-floor height (2 digits in cm or 3 digits in mm))	0
SHh	-	(Rear seat-to-floor height (2 digits in cm or 3 digits in mm))	0
SHD	-	(Front and rear seat-to-floor height difference (2 digits in cm or 3 digits in mm))	0

(Frame colours)

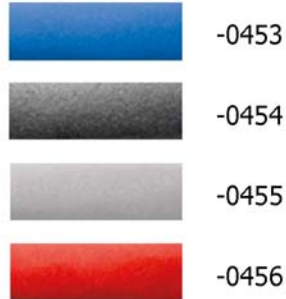


-0453	-	(Color code for attract blue frames)	0
-0454	-	(Color code for attract black frames)	0
-0455	-	(Color code for attract grey frames)	0
-0456	-	(Color code for attract red frames)	0

(CHASSIS)

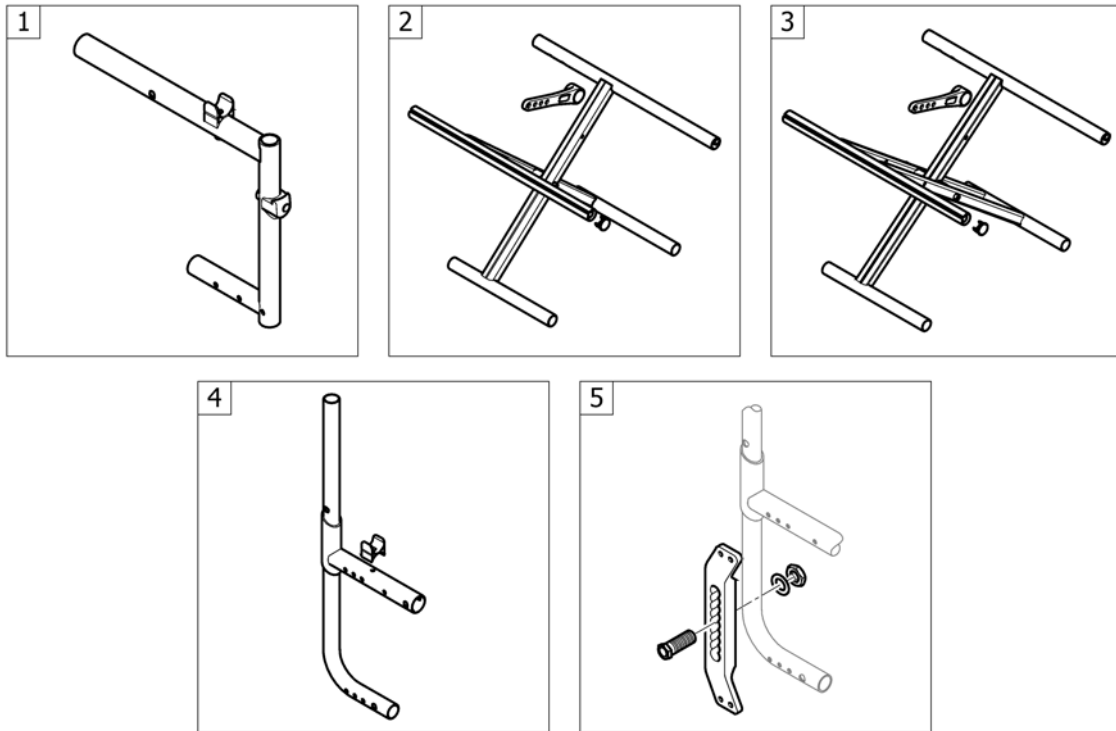

1	-	(Front frames standard)	1
2	-	(Cross brace single, SB36-42)	1
3	-	(Cross brace double, SB44-50)	1
4	-	(Rear frames)	1
5	-	(Rear wheel fixation)	1

(Front frames standard)

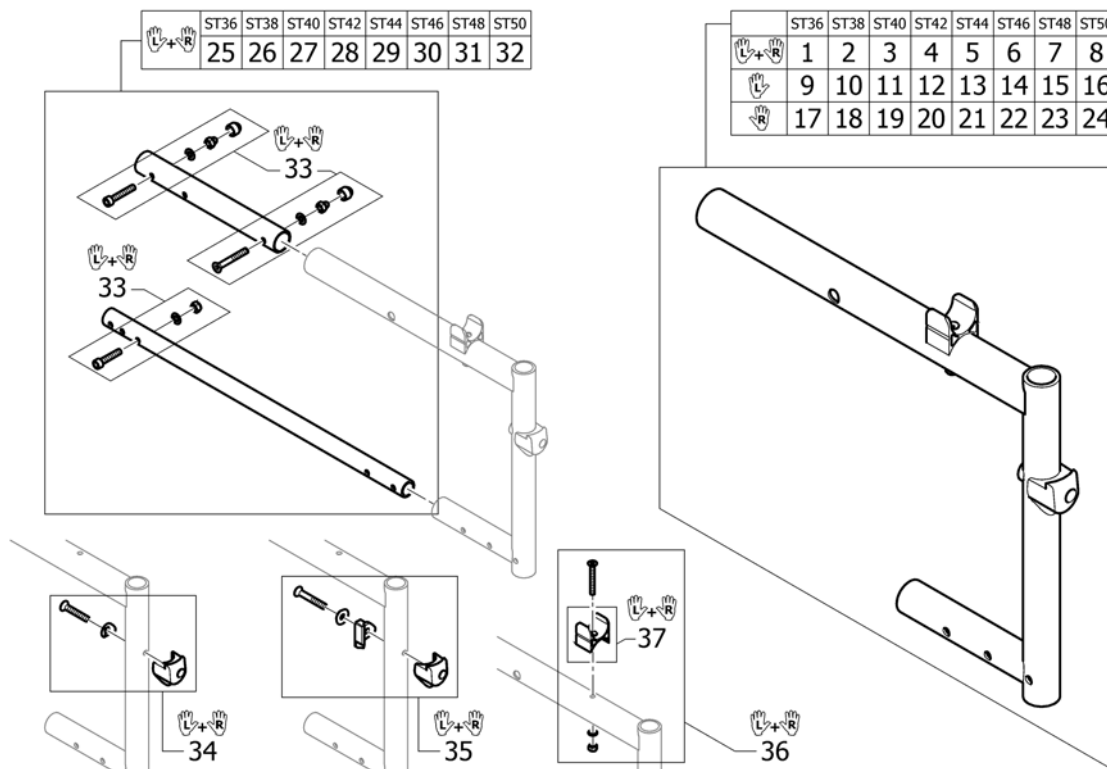

1	SP1602872-0453	(Front frame ST36, attract blue, pair)	1
1	SP1602872-0454	(Front frame ST36, attract black, pair)	1
1	SP1602872-0455	(Front frame ST36, attract grey, pair)	1
1	SP1602872-0456	(Front frame ST36, attract red, pair)	1
2	SP1602873-0453	(Front frame ST38, attract blue, pair)	1
2	SP1602873-0454	(Front frame ST38, attract black, pair)	1
2	SP1602873-0455	(Front frame ST38, attract grey, pair)	1
2	SP1602873-0456	(Front frame ST38, attract red, pair)	1
3	SP1602942-0453	(Front frame ST40, attract blue, pair)	1
3	SP1602942-0454	(Front frame ST40, attract black, pair)	1
3	SP1602942-0455	(Front frame ST40, attract grey, pair)	1
3	SP1602942-0456	(Front frame ST40, attract red, pair)	1
4	SP1602964-0453	(Front frame ST42, attract blue, pair)	1
4	SP1602964-0454	(Front frame ST42, attract black, pair)	1
4	SP1602964-0455	(Front frame ST42, attract grey, pair)	1
4	SP1602964-0456	(Front frame ST42, attract red, pair)	1
5	SP1602965-0453	(Front frame ST44, attract blue, pair)	1
5	SP1602965-0454	(Front frame ST44, attract black, pair)	1
5	SP1602965-0455	(Front frame ST44, attract grey, pair)	1
5	SP1602965-0456	(Front frame ST44, attract red, pair)	1
6	SP1602966-0453	(Front frame ST46, attract blue, pair)	1
6	SP1602966-0454	(Front frame ST46, attract black, pair)	1
6	SP1602966-0455	(Front frame ST46, attract grey, pair)	1
6	SP1602966-0456	(Front frame ST46, attract red, pair)	1
7	SP1602993-0453	(Front frame ST48, attract blue, pair)	1
7	SP1602993-0454	(Front frame ST48, attract black, pair)	1
7	SP1602993-0455	(Front frame ST48, attract grey, pair)	1
7	SP1602993-0456	(Front frame ST48, attract red, pair)	1
8	SP1602994-0453	(Front frame ST50, attract blue, pair)	1
8	SP1602994-0454	(Front frame ST50, attract black, pair)	1
8	SP1602994-0455	(Front frame ST50, attract grey, pair)	1


(Front frames standard)

8	SP1602994-0456	(Front frame ST50, attract red, pair)	1
9	SP1602995-0453	(Front frame left ST36, attract blue)	1
9	SP1602995-0454	(Front frame left ST36, attract black)	1
9	SP1602995-0455	(Front frame left ST36, attract grey)	1
9	SP1602995-0456	(Front frame left ST36, attract red)	1
10	SP1602996-0453	(Front frame left ST38, attract blue)	1
10	SP1602996-0454	(Front frame left ST38, attract black)	1
10	SP1602996-0455	(Front frame left ST38, attract grey)	1
10	SP1602996-0456	(Front frame left ST38, attract red)	1
11	SP1602997-0453	(Front frame left ST40, attract blue)	1
11	SP1602997-0454	(Front frame left ST40, attract black)	1
11	SP1602997-0455	(Front frame left ST40, attract grey)	1
11	SP1602997-0456	(Front frame left ST40, attract red)	1
12	SP1603008-0453	(Front frame left ST42, attract blue)	1
12	SP1603008-0454	(Front frame left ST42, attract black)	1
12	SP1603008-0455	(Front frame left ST42, attract grey)	1
12	SP1603008-0456	(Front frame left ST42, attract red)	1
13	SP1603009-0453	(Front frame left ST44, attract blue)	1
13	SP1603009-0454	(Front frame left ST44, attract black)	1
13	SP1603009-0455	(Front frame left ST44, attract grey)	1
13	SP1603009-0456	(Front frame left ST44, attract red)	1
14	SP1603010-0453	(Front frame left ST46, attract blue)	1
14	SP1603010-0454	(Front frame left ST46, attract black)	1
14	SP1603010-0455	(Front frame left ST46, attract grey)	1
14	SP1603010-0456	(Front frame left ST46, attract red)	1
15	SP1603011-0453	(Front frame left ST48, attract blue)	1
15	SP1603011-0454	(Front frame left ST48, attract black)	1
15	SP1603011-0455	(Front frame left ST48, attract grey)	1
15	SP1603011-0456	(Front frame left ST48, attract red)	1
16	SP1603012-0453	(Front frame left ST50, attract blue)	1
16	SP1603012-0454	(Front frame left ST50, attract black)	1
16	SP1603012-0455	(Front frame left ST50, attract grey)	1
16	SP1603012-0456	(Front frame left ST50, attract red)	1
17	SP1603013-0453	(Front frame right ST36, attract blue)	1
17	SP1603013-0454	(Front frame right ST36, attract black)	1
17	SP1603013-0455	(Front frame right ST36, attract grey)	1
17	SP1603013-0456	(Front frame right ST36, attract red)	1
18	SP1603014-0453	(Front frame right ST38, attract blue)	1
18	SP1603014-0454	(Front frame right ST38, attract black)	1
18	SP1603014-0455	(Front frame right ST38, attract grey)	1
18	SP1603014-0456	(Front frame right ST38, attract red)	1
19	SP1603015-0453	(Front frame right ST40, attract blue)	1
19	SP1603015-0454	(Front frame right ST40, attract black)	1
19	SP1603015-0455	(Front frame right ST40, attract grey)	1
19	SP1603015-0456	(Front frame right ST40, attract red)	1
20	SP1603016-0453	(Front frame right ST42, attract blue)	1
20	SP1603016-0454	(Front frame right ST42, attract black)	1
20	SP1603016-0455	(Front frame right ST42, attract grey)	1
20	SP1603016-0456	(Front frame right ST42, attract red)	1
21	SP1603017-0453	(Front frame right ST44, attract blue)	1
21	SP1603017-0454	(Front frame right ST44, attract black)	1
21	SP1603017-0455	(Front frame right ST44, attract grey)	1
21	SP1603017-0456	(Front frame right ST44, attract red)	1
22	SP1603018-0453	(Front frame right ST46, attract blue)	1
22	SP1603018-0454	(Front frame right ST46, attract black)	1

(Front frames standard)

22	SP1603018-0455	(Front frame right ST46, attract grey)	1
22	SP1603018-0456	(Front frame right ST46, attract red)	1
23	SP1603019-0453	(Front frame right ST48, attract blue)	1
23	SP1603019-0454	(Front frame right ST48, attract black)	1
23	SP1603019-0455	(Front frame right ST48, attract grey)	1
23	SP1603019-0456	(Front frame right ST48, attract red)	1
24	SP1603020-0453	(Front frame right ST50, attract blue)	1
24	SP1603020-0454	(Front frame right ST50, attract black)	1
24	SP1603020-0455	(Front frame right ST50, attract grey)	1
24	SP1603020-0456	(Front frame right ST50, attract red)	1
25	SP1603021	(Frames connecting tubes kit, ST36)	1
26	SP1603022	(Frames connecting tubes kit, ST38)	1
27	SP1603023	(Frames connecting tubes kit, ST40)	1
28	SP1603386	(Frames connecting tubes kit, ST42)	1
29	SP1603387	(Frames connecting tubes kit, ST44)	1
30	SP1603388	(Frames connecting tubes kit, ST46)	1
31	SP1603389	(Frames connecting tubes kit, ST48)	1
32	SP1603390	(Frames connecting tubes kit, ST50)	1
33	SP1603391	(Screws kit to frames connecting tubes)	1
34	SP1603392	(Position mechanism for swing-away, kit, pair)	1
35	SP1603777	(Rotating limiter for swing-away, kit, pair)	1
36	SP1603774	(Seat lock kit, complete)	1
37	SP1603393	(Seat lock kit)	1

(Cross brace single, SB36-42)

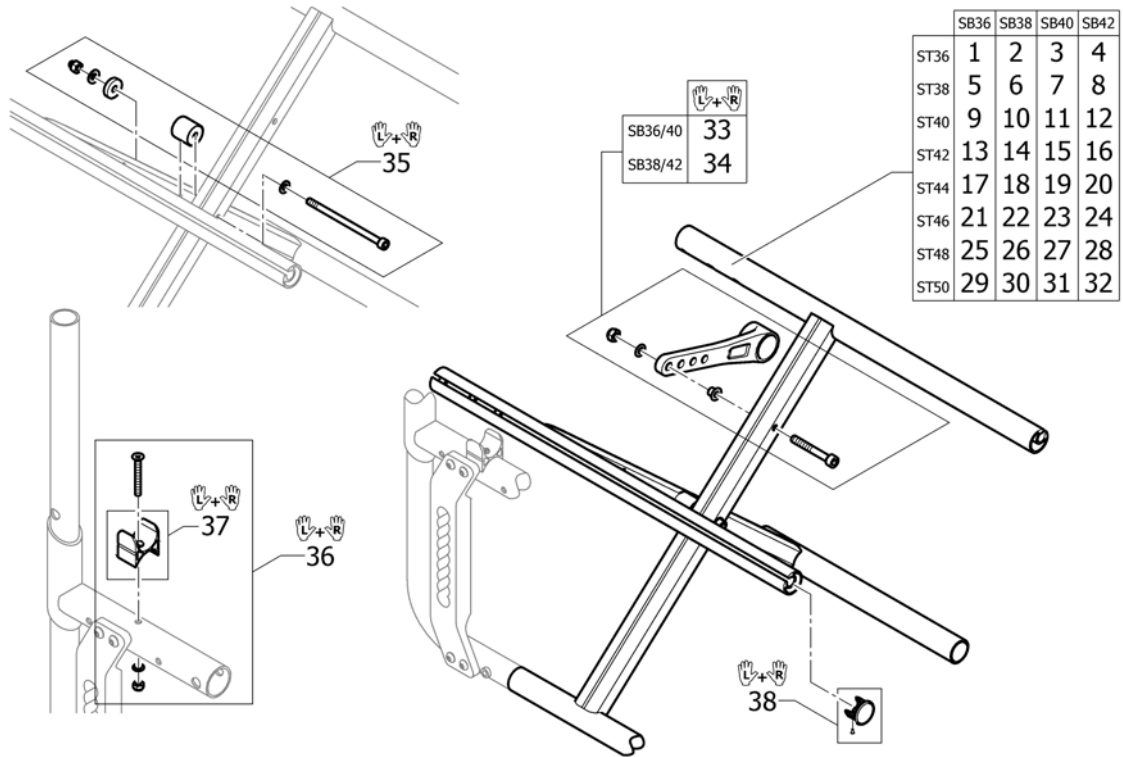


1	SP1603394-0454	(Cross-brace single, SB36 ST36, attract black)	1
2	SP1603402-0454	(Cross-brace single, SB38 ST36, attract black)	1
3	SP1603410-0454	(Cross-brace single, SB40 ST36, attract black)	1
4	SP1603512-0454	(Cross-brace single, SB42 ST36, attract black)	1
5	SP1603395-0454	(Cross-brace single, SB36 ST38, attract black)	1
6	SP1603403-0454	(Cross-brace single, SB38 ST38, attract black)	1
7	SP1603411-0454	(Cross-brace single, SB40 ST38, attract black)	1
8	SP1603545-0454	(Cross-brace single, SB42 ST38, attract black)	1
9	SP1603396-0454	(Cross-brace single, SB36 ST40, attract black)	1
10	SP1603404-0454	(Cross-brace single, SB38 ST40, attract black)	1
11	SP1603412-0454	(Cross-brace single, SB40 ST40, attract black)	1
12	SP1603554-0454	(Cross-brace single, SB42 ST40, attract black)	1
13	SP1603397-0454	(Cross-brace single, SB36 ST42, attract black)	1
14	SP1603405-0454	(Cross-brace single, SB38 ST42, attract black)	1
15	SP1603413-0454	(Cross-brace single, SB40 ST42, attract black)	1
16	SP1603555-0454	(Cross-brace single, SB42 ST42, attract black)	1
17	SP1603398-0454	(Cross-brace single, SB36 ST44, attract black)	1
18	SP1603406-0454	(Cross-brace single, SB38 ST44, attract black)	1
19	SP1603414-0454	(Cross-brace single, SB40 ST44, attract black)	1
20	SP1603556-0454	(Cross-brace single, SB42 ST44, attract black)	1
21	SP1603399-0454	(Cross-brace single, SB36 ST46, attract black)	1
22	SP1603407-0454	(Cross-brace single, SB38 ST46, attract black)	1
23	SP1603415-0454	(Cross-brace single, SB40 ST46, attract black)	1
24	SP1603557-0454	(Cross-brace single, SB42 ST46, attract black)	1
25	SP1603400-0454	(Cross-brace single, SB36 ST48, attract black)	1
26	SP1603408-0454	(Cross-brace single, SB38 ST48, attract black)	1
27	SP1603503-0454	(Cross-brace single, SB40 ST48, attract black)	1
28	SP1603401-0454	(Cross-brace single, SB36 ST50, attract black)	1
29	SP1603409-0454	(Cross-brace single, SB38 ST50, attract black)	1
30	SP1603504-0454	(Cross-brace single, SB40 ST50, attract black)	1
31	SP1603558-0454	(Cross-brace single, SB42 ST48, attract black)	1

Küschall® Compact attract | (CHASSIS)
(Cross brace single, SB36-42)

32	SP1603559-0454	(Cross-brace single, SB42 ST50, attract black)	1
33	SP1603751	(Retaining lever SB36/40/44/48, pair)	1
34	SP1603752	(Retaining lever SB38/42/46/50, pair)	1
35	SP1603753	(Screw kit to Single crossbrace)	1
36	SP1603774	(Seat lock kit, complete)	1
37	SP1603393	(Seat lock kit)	1
38	SP1603765	(Cross-brace plug kit, plastic)	1

(Cross brace double, SB44-50)

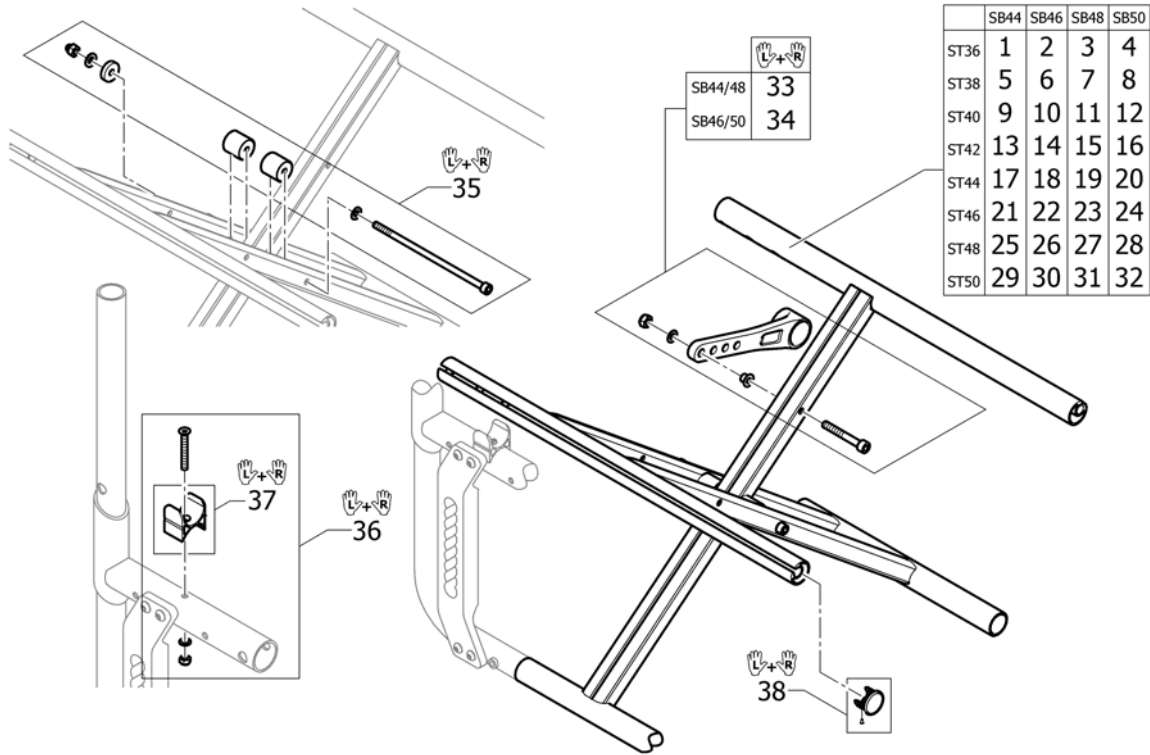


1	SP1603560-0454	(Cross-brace triplex, SB44 ST36, attract black)	1
2	SP1603568-0454	(Cross-brace triplex, SB46 ST36, attract black)	1
3	SP1603625-0454	(Cross-brace triplex, SB48 ST36, attract black)	1
4	SP1603743-0454	(Cross-brace triplex, SB50 ST36, attract black)	1
5	SP1603561-0454	(Cross-brace triplex, SB44 ST38, attract black)	1
6	SP1603569-0454	(Cross-brace triplex, SB46 ST38, attract black)	1
7	SP1603626-0454	(Cross-brace triplex, SB48 ST38, attract black)	1
8	SP1603744-0454	(Cross-brace triplex, SB50 ST38, attract black)	1
9	SP1603562-0454	(Cross-brace triplex, SB44 ST40, attract black)	1
10	SP1603619-0454	(Cross-brace triplex, SB46 ST40, attract black)	1
11	SP1603627-0454	(Cross-brace triplex, SB48 ST40, attract black)	1
12	SP1603745-0454	(Cross-brace triplex, SB50 ST40, attract black)	1
13	SP1603563-0454	(Cross-brace triplex, SB44 ST42, attract black)	1
14	SP1603620-0454	(Cross-brace triplex, SB46 ST42, attract black)	1
15	SP1603628-0454	(Cross-brace triplex, SB48 ST42, attract black)	1
16	SP1603746-0454	(Cross-brace triplex, SB50 ST42, attract black)	1
17	SP1603564-0454	(Cross-brace triplex, SB44 ST44, attract black)	1
18	SP1603621-0454	(Cross-brace triplex, SB46 ST44, attract black)	1
19	SP1603629-0454	(Cross-brace triplex, SB48 ST44, attract black)	1
20	SP1603747-0454	(Cross-brace triplex, SB50 ST44, attract black)	1
21	SP1603565-0454	(Cross-brace triplex, SB44 ST46, attract black)	1
22	SP1603622-0454	(Cross-brace triplex, SB46 ST46, attract black)	1
23	SP1603740-0454	(Cross-brace triplex, SB48 ST46, attract black)	1
24	SP1603748-0454	(Cross-brace triplex, SB50 ST46, attract black)	1
25	SP1603566-0454	(Cross-brace triplex, SB44 ST48, attract black)	1
26	SP1603623-0454	(Cross-brace triplex, SB46 ST48, attract black)	1
27	SP1603741-0454	(Cross-brace triplex, SB48 ST48, attract black)	1
28	SP1603749-0454	(Cross-brace triplex, SB50 ST48, attract black)	1
29	SP1603567-0454	(Cross-brace triplex, SB44 ST50, attract black)	1
30	SP1603624-0454	(Cross-brace triplex, SB46 ST50, attract black)	1
31	SP1603742-0454	(Cross-brace triplex, SB48 ST50, attract black)	1

Küschall® Compact attract | (CHASSIS)
(Cross brace double, SB44-50)


32	SP1603750-0454	(Cross-brace triplex, SB50 ST50, attract black)	1
33	SP1603751	(Retaining lever SB36/40/44/48, pair)	1
34	SP1603752	(Retaining lever SB38/42/46/50, pair)	1
35	SP1603764	(Screw kit to triplex crossbrace)	1
36	SP1603774	(Seat lock kit, complete)	1
37	SP1603393	(Seat lock kit)	1
38	SP1603765	(Cross-brace plug kit, plastic)	1

(Rear frames)

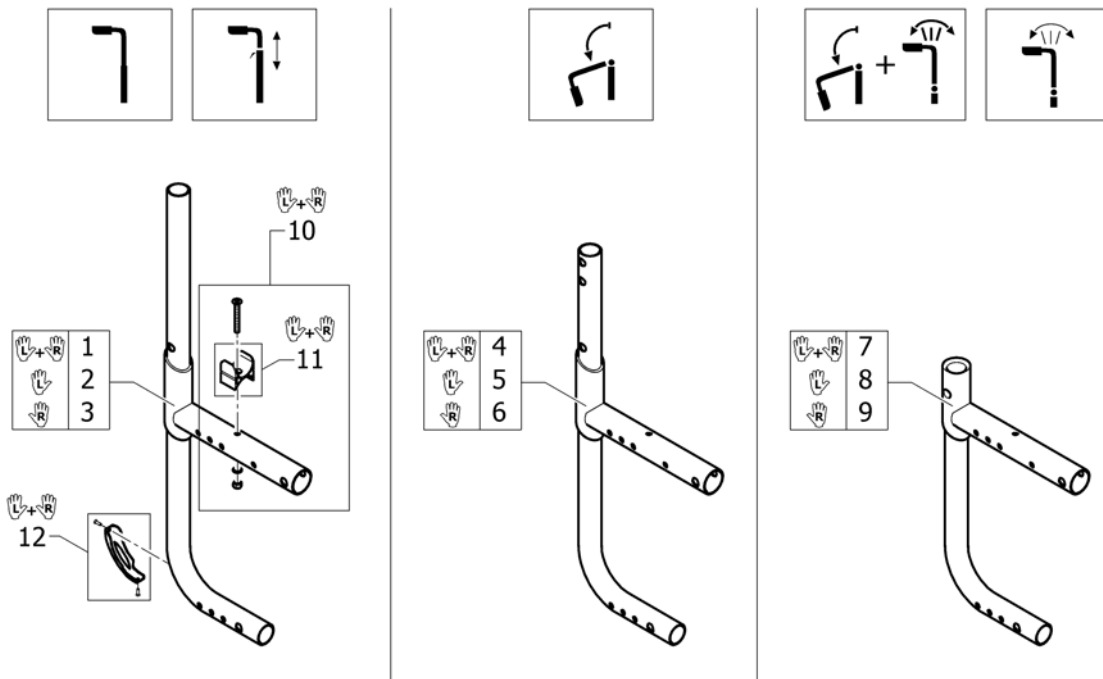




1	SP1603766-0454	(Rear frame, attract black, Pair)		1
2	SP1603767-0454	(Rear frame left, attract black)		1
3	SP1603768-0454	(Rear frame right, attract black)		1
4	SP1606530-0454	(Rear frame medium to foldable back, attract black, Pair)	01.07.2016 →	1
5	SP1606531-0454	(Rear frame left medium to foldable attract black)	01.07.2016 →	1
6	SP1606532-0454	(Rear frame right medium to foldable attract black)	01.07.2016 →	1
7	SP1603769-0454	(Rear frame short to angle adjustable back, attract black, Pair)		1
8	SP1603770-0454	(Rear frame left short to angle adjustable attract black, matt)		1
9	SP1603771-0454	(Rear frame right short to angle adjustable attract black, matt)		1
10	SP1603774	(Seat lock kit, complete)		1
11	SP1603393	(Seat lock kit)		1
12	1498774	BAKRAMME BESKYTTELSE KUSCHALL		1

(Rear wheel fixation)

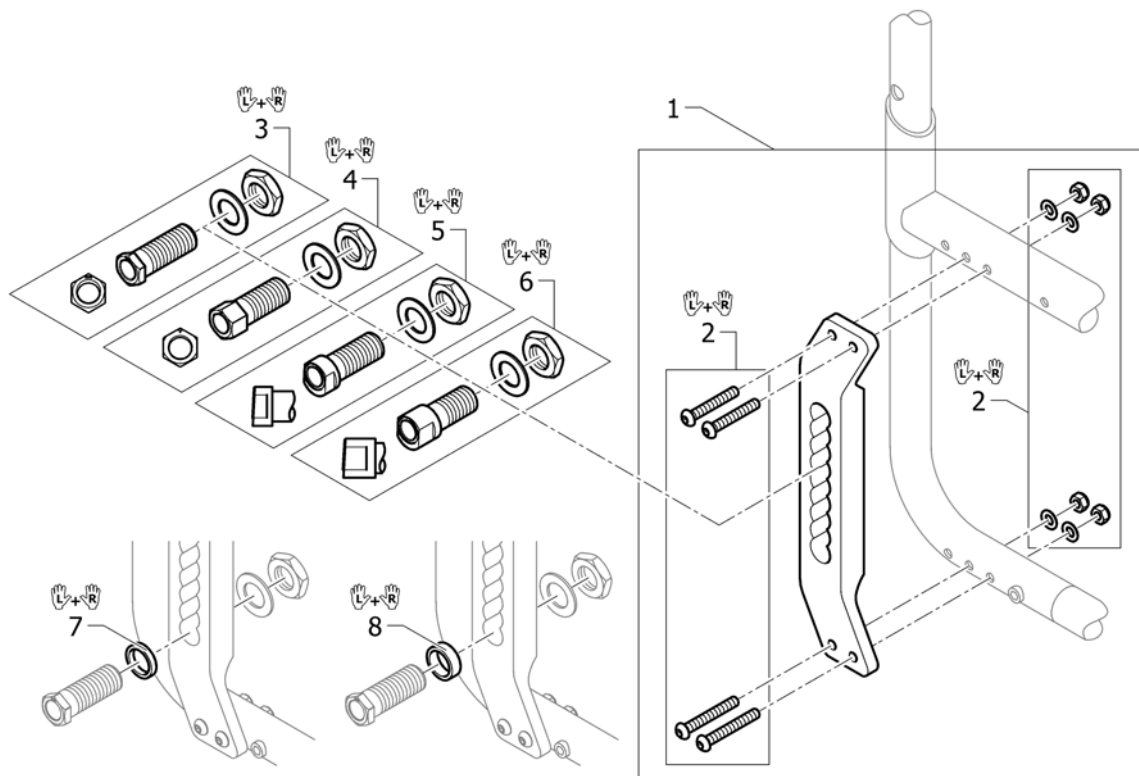


1	SP1603772	(Vertical brace, complete)	1
2	SP1603773	(Fixation kit for Vertical brace, pair)	1
3	SP1589279	(Adapter sleeve kit 2°, head short, pair)	1
4	SP1603776	(Adapter sleeve kit 1°, head long, pair)	1
5	SP1589282	(Adapter sleeve kit 4°, head short, pair)	1
6	SP1589281	(Adapter sleeve kit 4°, head long, pair)	1
7	SP1603775	(Spacer L=4mm for adapter sleeve, pair)	1
8	SP1589284	(Spacer L=7mm for adapter sleeve, pair)	1

(SEAT)

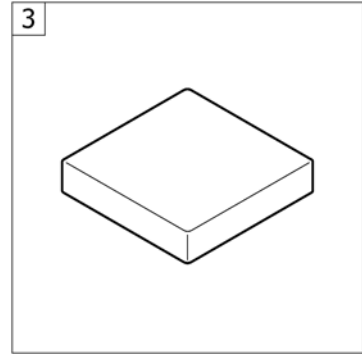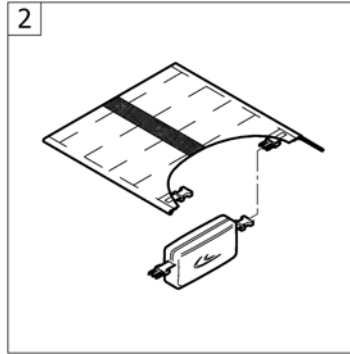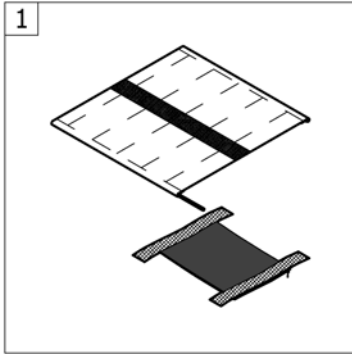

1	-	(Seat cover with 1 Velcro band (soft band))	1
2	-	(Seat cover with 1 Velcro band (soft band) for seat bag)	1
3	-	(Seat cushion standard)	1

(Seat cover with 1 Velcro band (soft band))

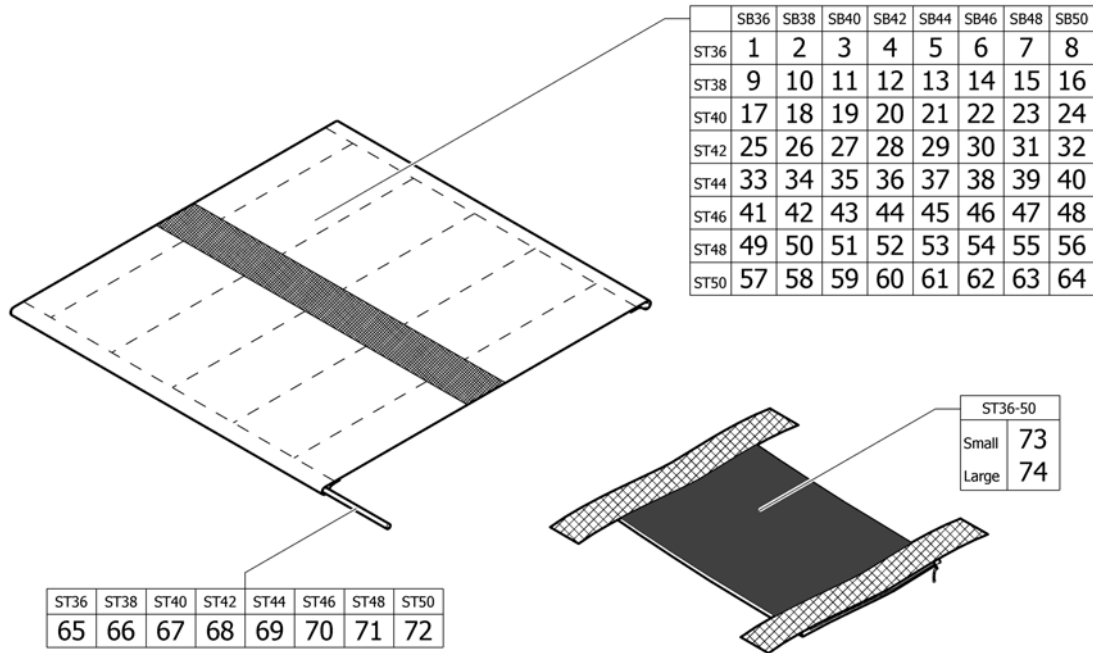


1	1540208	(Seat cover SB36 ST36 with 1 Velcro band (soft band))	1
2	1540209	(Seat cover SB38 ST36 with 1 Velcro band (soft band))	1
3	1540210	(Seat cover SB40 ST36 with 1 Velcro band (soft band))	1
4	1540211	(Seat cover SB42 ST36 with 1 Velcro band (soft band))	1
5	1540212	(Seat cover SB44 ST36 with 1 Velcro band (soft band))	1
6	1540213	(Seat cover SB46 ST36 with 1 Velcro band (soft band))	1
7	1540214	(Seat cover SB48 ST36 with 1 Velcro band (soft band))	1
8	1540215	(Seat cover SB50 ST36 with 1 Velcro band (soft band))	1
9	1540220	(Seat cover SB36 ST38 with 1 Velcro band (soft band))	1
10	1540221	(Seat cover SB38 ST38 with 1 Velcro band (soft band))	1
11	1540222	(Seat cover SB40 ST38 with 1 Velcro band (soft band))	1
12	1540223	(Seat cover SB42 ST38 with 1 Velcro band (soft band))	1
13	1540224	(Seat cover SB44 ST38 with 1 Velcro band (soft band))	1
14	1540225	(Seat cover SB46 ST38 with 1 Velcro band (soft band))	1
15	1540226	(Seat cover SB48 ST38 with 1 Velcro band (soft band))	1
16	1540227	(Seat cover SB50 ST38 with 1 Velcro band (soft band))	1
17	1540232	(Seat cover SB36 ST40 with 1 Velcro band (soft band))	1
18	1540233	(Seat cover SB38 ST40 with 1 Velcro band (soft band))	1
19	1540234	(Seat cover SB40 ST40 with 1 Velcro band (soft band))	1
20	1540235	(Seat cover SB42 ST40 with 1 Velcro band (soft band))	1
21	1540236	(Seat cover SB44 ST40 with 1 Velcro band (soft band))	1
22	1540237	(Seat cover SB46 ST40 with 1 Velcro band (soft band))	1
23	1540238	(Seat cover SB48 ST40 with 1 Velcro band (soft band))	1
24	1540239	(Seat cover SB50 ST40 with 1 Velcro band (soft band))	1
25	1540244	(Seat cover SB36 ST42 with 1 Velcro band (soft band))	1
26	1540245	(Seat cover SB38 ST42 with 1 Velcro band (soft band))	1
27	1540246	(Seat cover SB40 ST42 with 1 Velcro band (soft band))	1
28	1540247	(Seat cover SB42 ST42 with 1 Velcro band (soft band))	1
29	1540248	(Seat cover SB44 ST42 with 1 Velcro band (soft band))	1
30	1540249	(Seat cover SB46 ST42 with 1 Velcro band (soft band))	1
31	1540250	(Seat cover SB48 ST42 with 1 Velcro band (soft band))	1

(Seat cover with 1 Velcro band (soft band))

32	1540251	(Seat cover SB50 ST42 with 1 Velcro band (soft band))	1
33	1540256	(Seat cover SB36 ST44 with 1 Velcro band (soft band))	1
34	1540257	(Seat cover SB38 ST44 with 1 Velcro band (soft band))	1
35	1540258	(Seat cover SB40 ST44 with 1 Velcro band (soft band))	1
36	1540259	(Seat cover SB42 ST44 with 1 Velcro band (soft band))	1
37	1540682	(Seat cover SB44 ST44 with 1 Velcro band (soft band))	1
38	1540683	(Seat cover SB46 ST44 with 1 Velcro band (soft band))	1
39	1540684	(Seat cover SB48 ST44 with 1 Velcro band (soft band))	1
40	1540685	(Seat cover SB50 ST44 with 1 Velcro band (soft band))	1
41	1540690	(Seat cover SB36 ST46 with 1 Velcro band (soft band))	1
42	1540691	(Seat cover SB38 ST46 with 1 Velcro band (soft band))	1
43	1540692	(Seat cover SB40 ST46 with 1 Velcro band (soft band))	1
44	1540693	(Seat cover SB42 ST46 with 1 Velcro band (soft band))	1
45	1540694	(Seat cover SB44 ST46 with 1 Velcro band (soft band))	1
46	1540695	(Seat cover SB46 ST46 with 1 Velcro band (soft band))	1
47	1540696	(Seat cover SB48 ST46 with 1 Velcro band (soft band))	1
48	1540697	(Seat cover SB50 ST46 with 1 Velcro band (soft band))	1
49	1540702	(Seat cover SB36 ST48 with 1 Velcro band (soft band))	1
50	1540703	(Seat cover SB38 ST48 with 1 Velcro band (soft band))	1
51	1540704	(Seat cover SB40 ST48 with 1 Velcro band (soft band))	1
52	1540705	(Seat cover SB42 ST48 with 1 Velcro band (soft band))	1
53	1540706	(Seat cover SB44 ST48 with 1 Velcro band (soft band))	1
54	1540707	(Seat cover SB46 ST48 with 1 Velcro band (soft band))	1
55	1540708	(Seat cover SB48 ST48 with 1 Velcro band (soft band))	1
56	1540709	(Seat cover SB50 ST48 with 1 Velcro band (soft band))	1
57	1540714	(Seat cover SB36 ST50 with 1 Velcro band (soft band))	1
58	1540715	(Seat cover SB38 ST50 with 1 Velcro band (soft band))	1
59	1540716	(Seat cover SB40 ST50 with 1 Velcro band (soft band))	1
60	1540717	(Seat cover SB42 ST50 with 1 Velcro band (soft band))	1
61	1540718	(Seat cover SB44 ST50 with 1 Velcro band (soft band))	1
62	1540719	(Seat cover SB46 ST50 with 1 Velcro band (soft band))	1
63	1540720	(Seat cover SB48 ST50 with 1 Velcro band (soft band))	1
64	1540721	(Seat cover SB50 ST50 with 1 Velcro band (soft band))	1
65	1564334	(Seat cover bar kit ST36)	1
66	1564336	(Seat cover bar kit ST38)	1
67	1564337	(Seat cover bar kit ST40)	1
68	1564338	(Seat cover bar kit ST42)	1
69	1564339	(Seat cover bar kit ST44)	1
70	1564341	(Seat cover bar kit ST46)	1
71	1564342	(Seat cover bar kit ST48)	1
72	1564343	(Seat cover bar kit ST50)	1
73	1547623	(Seat upholstery bag, small)	1
74	1547624	(Seat upholstery bag, big)	1


(Seat cover with 1 Velcro band (soft band) for seat bag)

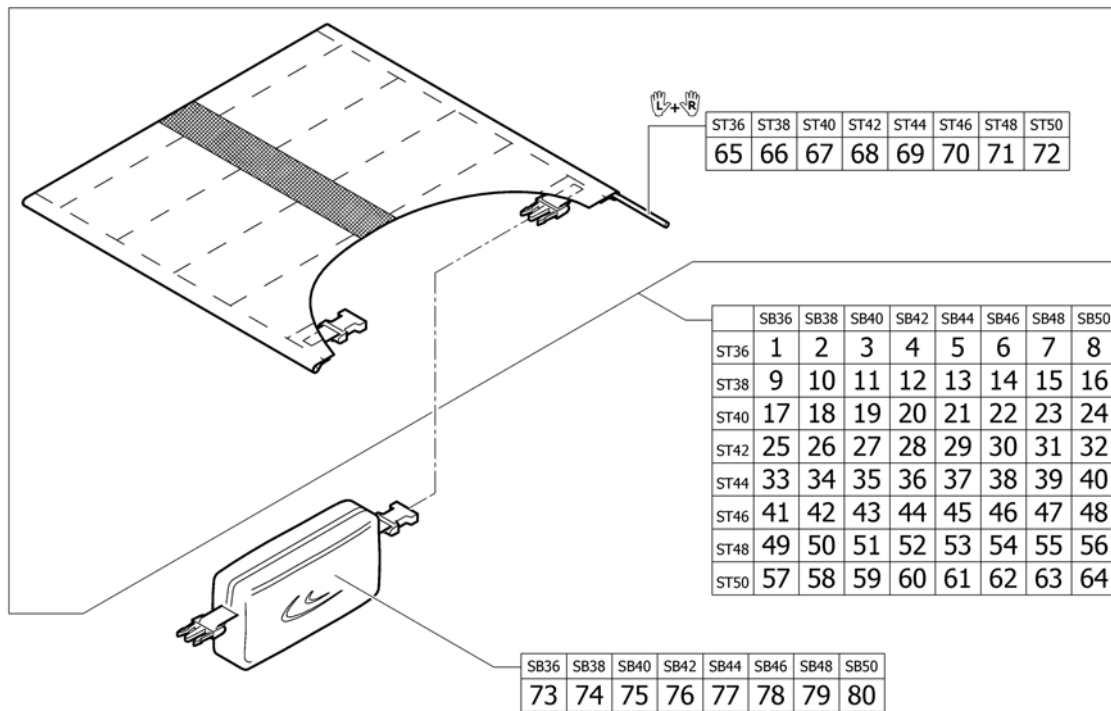

1	1576589	(Seat cover SB36 ST36 for seat bag with 1 Velcro band (soft band))	1
2	1576695	(Seat cover SB38 ST36 for seat bag with 1 Velcro band (soft band))	1
3	1576696	(Seat cover SB40 ST36 for seat bag with 1 Velcro band (soft band))	1
4	1576697	(Seat cover SB42 ST36 for seat bag with 1 Velcro band (soft band))	1
5	1576698	(Seat cover SB44 ST36 for seat bag with 1 Velcro band (soft band))	1
6	1576699	(Seat cover SB46 ST36 for seat bag with 1 Velcro band (soft band))	1
7	1576700	(Seat cover SB48 ST36 for seat bag with 1 Velcro band (soft band))	1
8	1576701	(Seat cover SB50 ST36 for seat bag with 1 Velcro band (soft band))	1
9	1576706	(Seat cover SB36 ST38 for seat bag with 1 Velcro band (soft band))	1
10	1576707	(Seat cover SB38 ST38 for seat bag with 1 Velcro band (soft band))	1
11	1576708	(Seat cover SB40 ST38 for seat bag with 1 Velcro band (soft band))	1
12	1576709	(Seat cover SB42 ST38 for seat bag with 1 Velcro band (soft band))	1
13	1576710	(Seat cover SB44 ST38 for seat bag with 1 Velcro band (soft band))	1
14	1577138	(Seat cover SB46 ST38 for seat bag with 1 Velcro band (soft band))	1
15	1577139	(Seat cover SB48 ST38 for seat bag with 1 Velcro band (soft band))	1
16	1577288	(Seat cover SB50 ST38 for seat bag with 1 Velcro band (soft band))	1
17	1577992	(Seat cover SB36 ST40 for seat bag with 1 Velcro band (soft band))	1
18	1578183	(Seat cover SB38 ST40 for seat bag with 1 Velcro band (soft band))	1
19	1578184	(Seat cover SB40 ST40 for seat bag with 1 Velcro band (soft band))	1
20	1578185	(Seat cover SB42 ST40 for seat bag with 1 Velcro band (soft band))	1
21	1578186	(Seat cover SB44 ST40 for seat bag with 1 Velcro band (soft band))	1
22	1578187	(Seat cover SB46 ST40 for seat bag with 1 Velcro band (soft band))	1
23	1578188	(Seat cover SB48 ST40 for seat bag with 1 Velcro band (soft band))	1
24	1578189	(Seat cover SB50 ST40 for seat bag with 1 Velcro band (soft band))	1
25	1578323	(Seat cover SB36 ST42 for seat bag with 1 Velcro band (soft band))	1
26	1578324	(Seat cover SB38 ST42 for seat bag with 1 Velcro band (soft band))	1
27	1578325	(Seat cover SB40 ST42 for seat bag with 1 Velcro band (soft band))	1
28	1578400	(Seat cover SB42 ST42 for seat bag with 1 Velcro band (soft band))	1
29	1578401	(Seat cover SB44 ST42 for seat bag with 1 Velcro band (soft band))	1
30	1578423	(Seat cover SB46 ST42 for seat bag with 1 Velcro band (soft band))	1
31	1578424	(Seat cover SB48 ST42 for seat bag with 1 Velcro band (soft band))	1

(Seat cover with 1 Velcro band (soft band) for seat bag)

32	1578425	(Seat cover SB50 ST42 for seat bag with 1 Velcro band (soft band))	1
33	1578430	(Seat cover SB36 ST44 for seat bag with 1 Velcro band (soft band))	1
34	1578618	(Seat cover SB38 ST44 for seat bag with 1 Velcro band (soft band))	1
35	1578619	(Seat cover SB40 ST44 for seat bag with 1 Velcro band (soft band))	1
36	1578620	(Seat cover SB42 ST44 for seat bag with 1 Velcro band (soft band))	1
37	1578621	(Seat cover SB44 ST44 for seat bag with 1 Velcro band (soft band))	1
38	1578622	(Seat cover SB46 ST44 for seat bag with 1 Velcro band (soft band))	1
39	1578623	(Seat cover SB48 ST44 for seat bag with 1 Velcro band (soft band))	1
40	1578624	(Seat cover SB50 ST44 for seat bag with 1 Velcro band (soft band))	1
41	1578629	(Seat cover SB36 ST46 for seat bag with 1 Velcro band (soft band))	1
42	1578630	(Seat cover SB38 ST46 for seat bag with 1 Velcro band (soft band))	1
43	1578631	(Seat cover SB40 ST46 for seat bag with 1 Velcro band (soft band))	1
44	1578632	(Seat cover SB42 ST46 for seat bag with 1 Velcro band (soft band))	1
45	1578633	(Seat cover SB44 ST46 for seat bag with 1 Velcro band (soft band))	1
46	1578634	(Seat cover SB46 ST46 for seat bag with 1 Velcro band (soft band))	1
47	1578635	(Seat cover SB48 ST46 for seat bag with 1 Velcro band (soft band))	1
48	1578636	(Seat cover SB50 ST46 for seat bag with 1 Velcro band (soft band))	1
49	1578641	(Seat cover SB36 ST48 for seat bag with 1 Velcro band (soft band))	1
50	1578642	(Seat cover SB38 ST48 for seat bag with 1 Velcro band (soft band))	1
51	1578643	(Seat cover SB40 ST48 for seat bag with 1 Velcro band (soft band))	1
52	1578644	(Seat cover SB42 ST48 for seat bag with 1 Velcro band (soft band))	1
53	1578645	(Seat cover SB44 ST48 for seat bag with 1 Velcro band (soft band))	1
54	1578646	(Seat cover SB46 ST48 for seat bag with 1 Velcro band (soft band))	1
55	1578647	(Seat cover SB48 ST48 for seat bag with 1 Velcro band (soft band))	1
56	1578648	(Seat cover SB50 ST48 for seat bag with 1 Velcro band (soft band))	1
57	1578653	(Seat cover SB36 ST50 for seat bag with 1 Velcro band (soft band))	1
58	1578654	(Seat cover SB38 ST50 for seat bag with 1 Velcro band (soft band))	1
59	1578655	(Seat cover SB40 ST50 for seat bag with 1 Velcro band (soft band))	1
60	1578656	(Seat cover SB42 ST50 for seat bag with 1 Velcro band (soft band))	1
61	1578657	(Seat cover SB44 ST50 for seat bag with 1 Velcro band (soft band))	1
62	1578658	(Seat cover SB46 ST50 for seat bag with 1 Velcro band (soft band))	1
63	1578659	(Seat cover SB48 ST50 for seat bag with 1 Velcro band (soft band))	1
64	1578660	(Seat cover SB50 ST50 for seat bag with 1 Velcro band (soft band))	1
65	1564334	(Seat cover bar kit ST36)	1
66	1564336	(Seat cover bar kit ST38)	1
67	1564337	(Seat cover bar kit ST40)	1
68	1564338	(Seat cover bar kit ST42)	1
69	1564339	(Seat cover bar kit ST44)	1
70	1564341	(Seat cover bar kit ST46)	1
71	1564342	(Seat cover bar kit ST48)	1
72	1564343	(Seat cover bar kit ST50)	1
73	1578788	(Seat bag SB36)	1
74	1578789	VESKE FOR SETE SB38 KUSCHALL	1
75	1578790	(Seat bag SB40)	1
76	1578791	(Seat bag SB42)	1
77	1578792	(Seat bag SB44)	1
78	1578793	(Seat bag SB46)	1
79	1578794	(Seat bag SB48)	1
80	1578795	(Seat bag SB50)	1

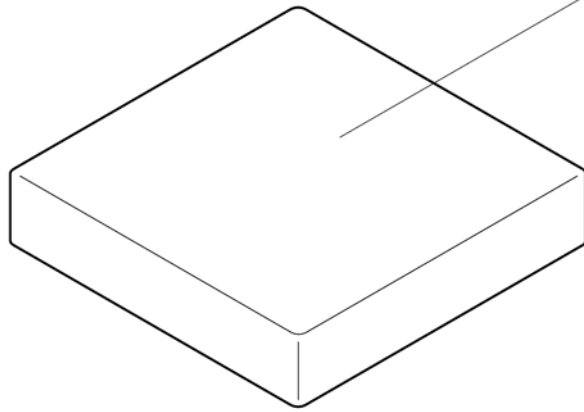


	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
ST36	1	2	3	4	5	6	7	8
ST38	2	9	10	11	12	13	14	15
ST40	3	10	16	17	18	19	20	21
ST42	4	11	17	22	23	24	25	26
ST44	5	12	18	23	27	28	29	30
ST82	6	13	19	24	28	31	32	33
ST48	7	14	20	25	29	32	34	35
ST50	8	15	21	26	30	33	35	36

1	1556335	SETEPUTE 36X36X5 KUSCHALL	1
2	1556336	PUTE SETE SB 36 SD38 KUSCHALL	1
3	1556337	SETEPUTE 360mm x 400mm KUSCHALL	1
4	1556338	(Seat cushion 360mm x 420mm)	1
5	1556339	(Seat cushion 360mm x 440mm)	1
6	1556340	(Seat cushion 360mm x 460mm)	1
7	1556341	(Seat cushion 360mm x 480mm)	1
8	1556342	(Seat cushion 360mm x 500mm)	1
9	1556344	SETEPUTE 380mm x 380mm KUSCHALL	1
10	1556345	SETEPUTE STD 380mm x 400mm	1
11	1556346	SETEPUTE 42/38/5 SORT KUSCHALL	1
12	1556347	(Seat cushion 380mm x 440mm)	1
13	1556348	(Seat cushion 380mm x 460mm)	1
14	1556349	(Seat cushion 380mm x 480mm)	1
15	1556350	(Seat cushion 380mm x 500mm)	1
16	1556352	SETEPUTE 40/40/5 SORT KUSCHALL	1
17	1556353	SETEPUTE 42/40/5 SORT KUSCHALL	1
18	1556354	SETEPUTE 40X44X5 SORT KUSCHALL	1
19	1556355	SETEPUTE 400mm x 460mm KUSCHALL	1
20	1556356	(Seat cushion 400mm x 480mm)	1
21	1556357	SETEPUTE 40X50 KUSCHALL	1
22	1556359	SETEPUTE 420mm x 420mm KUSCHALL	1
23	1556360	(Seat cushion 420mm x 440mm)	1
24	1556361	SETEPUTE 420mm x 460mm KUSCHALL	1
25	1556362	(Seat cushion 420mm x 480mm)	1
26	1556363	(Seat cushion 420mm x 500mm)	1
27	1556365	SETEPUTE 44/44/5 SORT KUSCHALL	1
28	1556366	SETEPUTE 46/44/5 SORT KUSCHALL	1
29	1556367	SETEPUTE 44/48/5 SORT KUSCHALL	1
30	1556368	(Seat cushion 440mm x 500mm)	1
31	1556370	SETEPUTE 46/46/5 SORT KUSCHALL	1


(Seat cushion standard)

32	1556371	(Seat cushion 460mm x 480mm)	1
33	1556372	(Seat cushion 460mm x 500mm)	1
34	1556374	SETEPUTE 48/48/5 SORT KUSCHALL	1
35	1556375	(Seat cushion 480mm x 500mm)	1
36	1556377	(Seat cushion 500mm x 500mm)	1

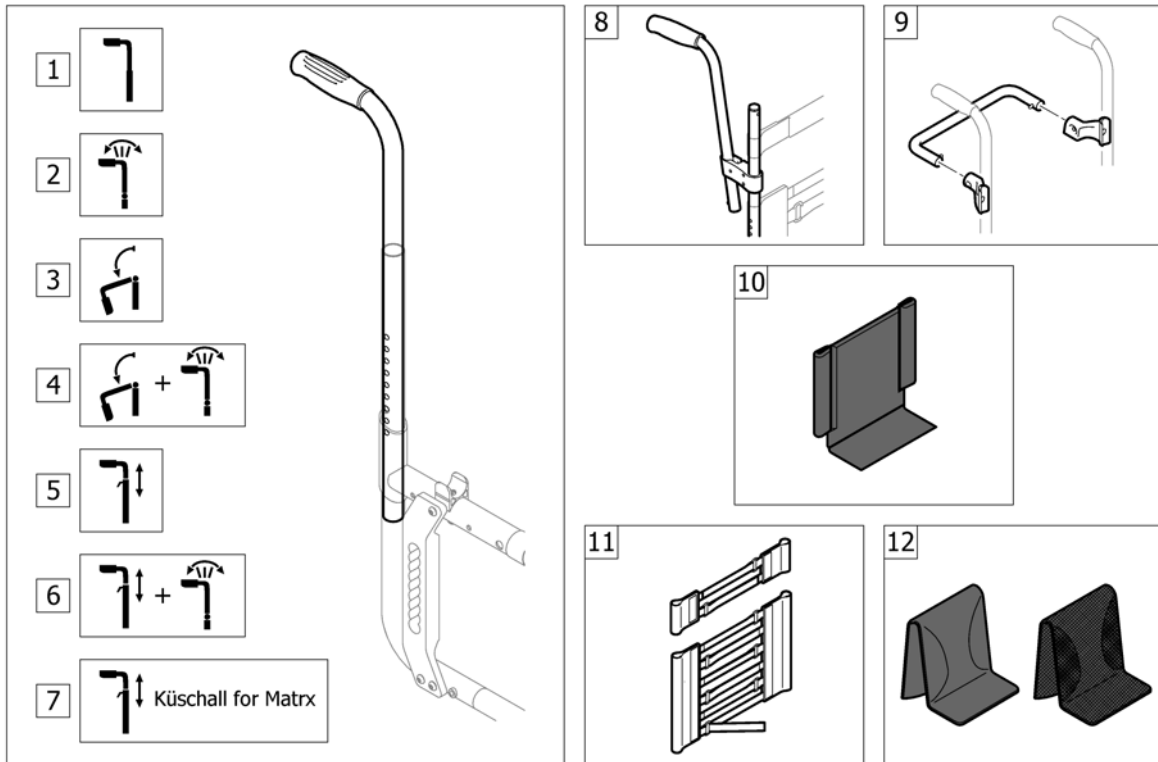


1	-	(Push handles for backrest standard)		1
2	-	(Push handles for backrest angle adjustable)		1
3	-	(Push handles for backrest foldable)	01.07.2016 →	1
4	-	(Push handles for backrest foldable and angle adjustable)	01.07.2016 →	1
5	-	(Push handles height adjustable)		1
6	-	(Push handles height adjustable for backrest angle adjustable)		1
7	-	(Küschall backrest parts for Matrix backs)		1
8	-	(Push handles, inclined)		1
9	-	(Backrest stabilization bar)		1
10	-	(Back cover, standard)		1
11	-	(Bands to adjustable backrest)		1
12	-	(Cover to adjustable backrest)		1

(Push handles for backrest standard)

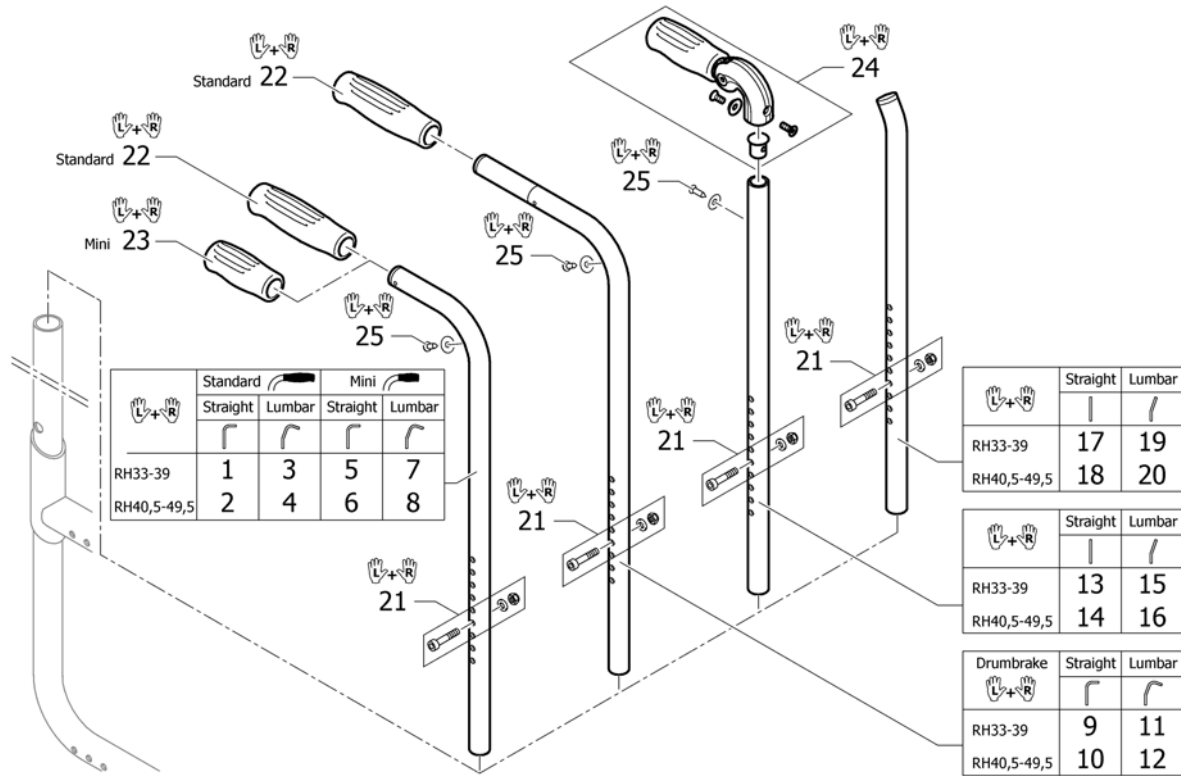


1	1540072	KJØREHÅNDTAK KORTE PAR KUSCHALL COMPACT	1
2	1540073	KJØREHÅNDTAK MEDIUM PAR KUSCHALL COMPACT	1
3	1540074	KJØREHÅNDTAK STD LUMBAR KORT	1
4	1540075	(Push handles standard lumbar medium, pair)	1
5	1540076	KJØREHÅNDTAK MINI KORT PAR	1
6	1540077	(Push handles mini, medium, pair)	1
7	1540078	(Push handles mini lumbar short, pair)	1
8	1540079	(Push handles mini lumbar medium, pair)	1
9	1540084	KJØREHÅNDTAK LEDBREMS KUSCHALL KORT PAR	1
10	1540085	(Push handles medium to drum brake, pair)	1
11	1540086	(Push handles lumbar short to drum brake, pair)	1
12	1540087	(Push handles lumbar medium to drum brake, pair)	1
13	1540080	(Backrest tube straight short to foldable push handles, pair)	1
14	1540081	(Backrest tube straight medium to foldable push handles, pair)	1
15	1540082	(Backrest tube lumbar short to foldable push handles, pair)	1
16	1540083	(Backrest tube lumbar medium to foldable push handles, pair)	1
17	1540089	(Backrest tubes L without push handles, pair)	1
18	1540090	RYGGRØR U/KJØREHÅNDTAK MEDIUM LIFE KUSCHALL	1
19	1540091	(Backrest tubes lumbar L without push handles, pair)	1
20	1540092	(Backrest tubes lumbar XL without push handles, pair)	1
21	1540088	FESTE KJØREHÅNDTAK LEDSAGERBREMS KUSCHALL	1
22	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1
23	1525126	KJØREHÅNDTAK MINI SORT PLAST KUSCHALL	1
24	1507733	KJØREHÅNDTAK FOLDBARE KUSCHALL CO PAR	1
25	1432083	FESTEKIT FOR RYGGBÅND CHAMPION KUSCHALL	1

(Push handles for backrest angle adjustable)

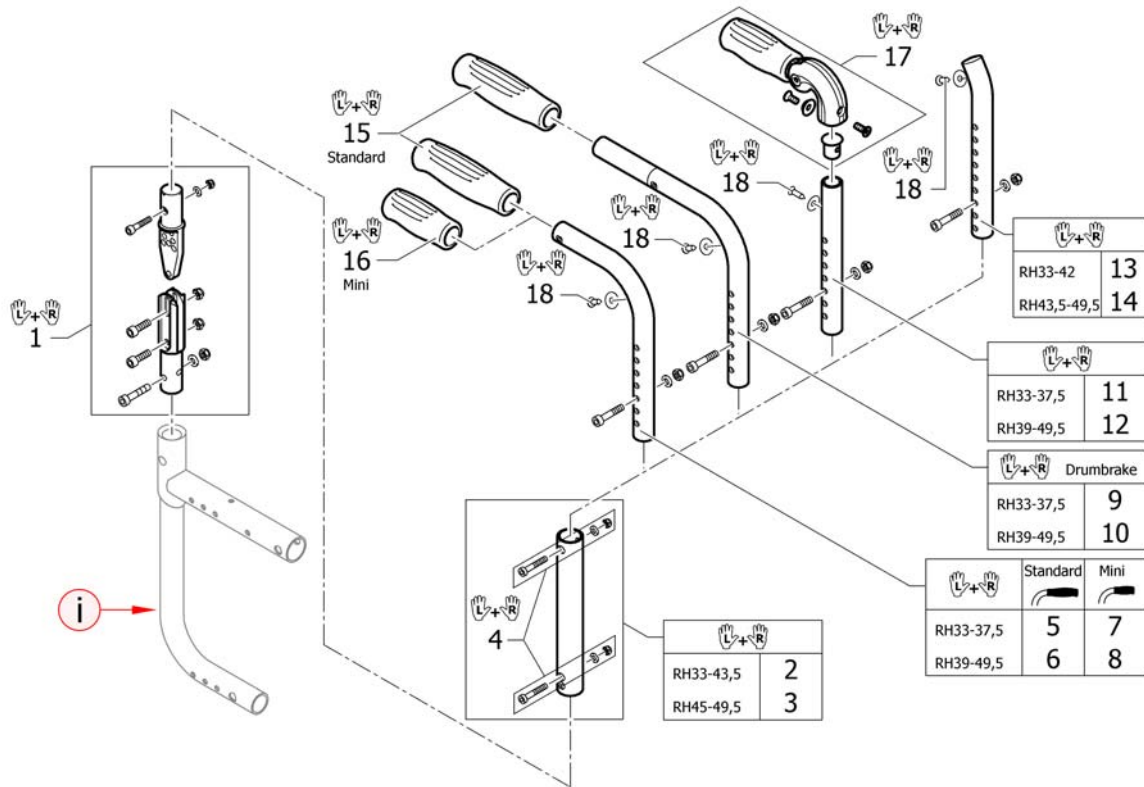


i	-	(A special rear frame is necessary. Please see chapter CHASSIS.)	0
1	1540094	RYGG VINKEL SETT	1
2	1540063	RYGGRØR VREG RYGG RH30-51	1
3	1540095	RYGGRØR VREG LANG SETT	1
4	1540065	(Screw kit for telescopic tube)	1
5	1540055	(Push handles standard, XS, pair)	1
6	1540056	(Push handles standard, S, pair)	1
7	1540069	(Push handles mini, XS, pair)	1
8	1540070	(Push handles mini, S, pair)	1
9	1540057	(Push handles XS to drum brake, pair)	1
10	1540058	(Push handles S to drum brake, pair)	1
11	1540059	(Backrest tube straight XS to foldable push handles, pair)	1
12	1540060	(Backrest tube straight S to foldable push handles, pair)	1
13	1540061	RYGGRØR RH30-37,5 UTEN KJØREHÅNDTAK KORTE	1
14	1540062	(Backrest tubes M without push handles, pair)	1
15	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1
16	1525126	KJØREHÅNDTAK MINI SORT PLAST KUSCHALL	1
17	1507733	KJØREHÅNDTAK FOLDBARE KUSCHALL CO PAR	1
18	1432083	FESTEKIT FOR RYGGBÅND CHAMPION KUSCHALL	1

(Push handles for backrest foldable)

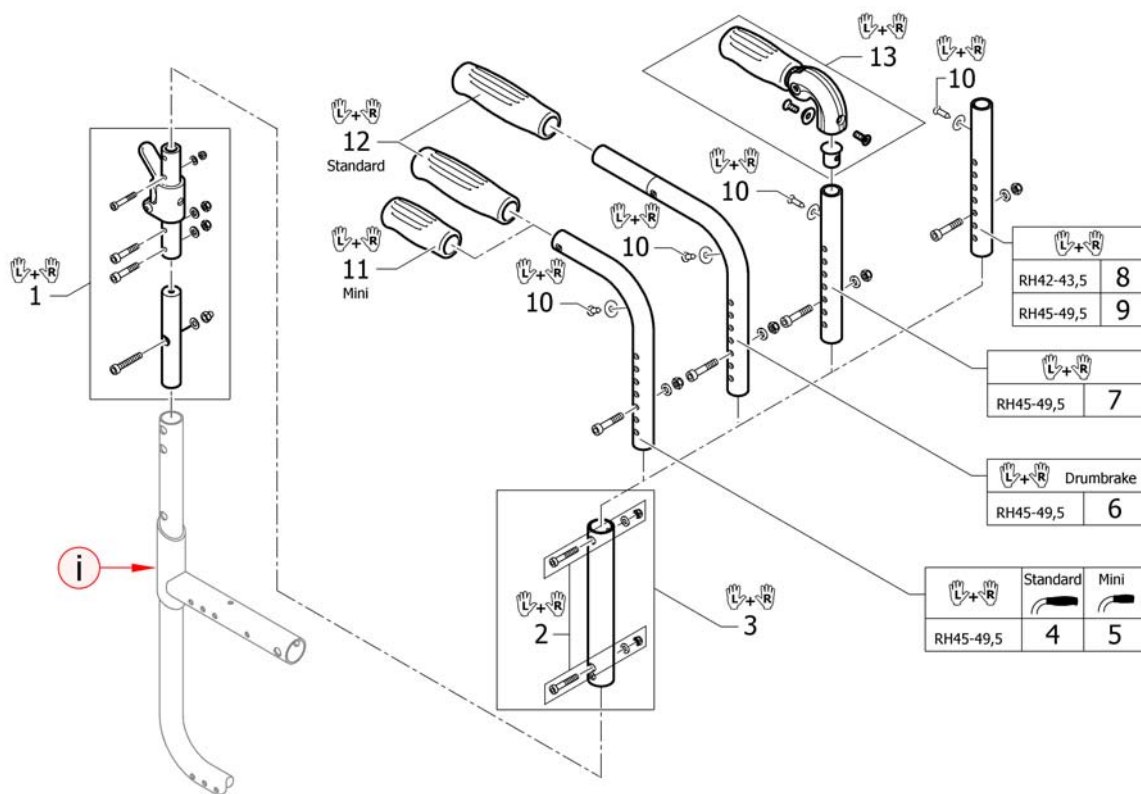


i	-	(A special rear frame is necessary. Please see chapter CHASSIS.)	0
1	SP1606525	(Hinge kit to backrest foldable, pair)	1
2	1509134	(Screw kit for telescopic tube)	1
3	1438209	RYGGRØR TELESKOPIK KIT 34,5 - 43,5 KUSCHALL CO	1
4	1507739	HÅNDTAK STD XS PAR	1
5	1525342	(Push handles mini, XS, pair)	1
6	1507743	(Push handles XS to drum brake, pair)	1
7	1507735	RYGGRØR RAK KJØREHÅNDTAK FOLDBARE XS KUSCHALL CO	1
8	SP1606526	(Backrest tube straight XXS for backrest foldable, pair)	1
9	SP1606527	(Backrest tube straight XS for backrest foldable, pair)	1
10	1432083	FESTEKIT FOR RYGGBÅND CHAMPION KÜSCHALL	1
11	1525126	KJØREHÅNDTAK MINI SORT PLAST KUSCHALL	1
12	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1
13	1507733	KJØREHÅNDTAK FOLDBARE KUSCHALL CO PAR	1

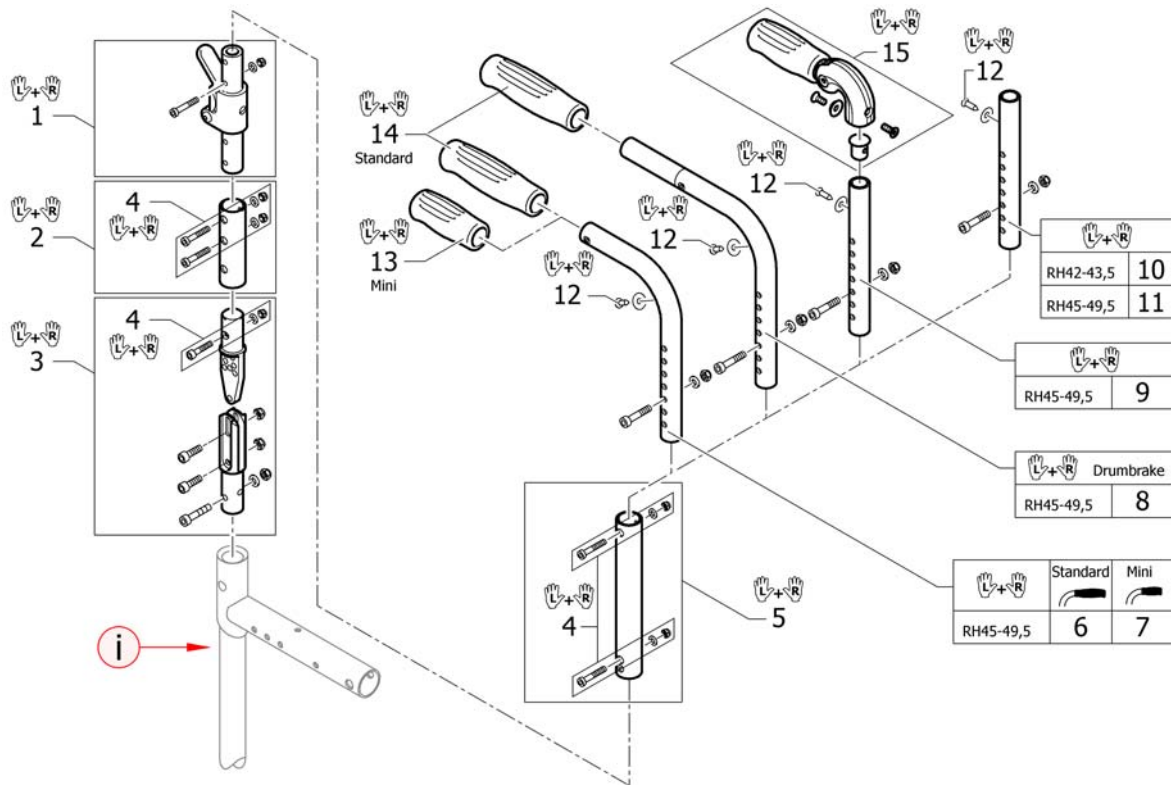(Push handles for backrest foldable and angle adjustable)


i	-	(A special rear frame is necessary. Please see chapter CHASSIS.)	0
1	1507749	LEDD TIL FELLBAR OG VREG RYGG PAR CO	1
2	1507748	KOBLINGSRØR FELLBAR OG VREG RYGG PAR CO	1
3	1540094	RYGG VINKEL SETT	1
4	1509135	SKRUE KIT TIL KOBLINGSRØR VREG OG FELL	1
5	1438209	RYGGRØR TELESKOPISK KIT 34,5 - 43,5 KUSCHALL CO	1
6	1507739	HÅNDTAK STD XS PAR	1
7	1525342	(Push handles mini, XS, pair)	1
8	1507743	(Push handles XS to drum brake, pair)	1
9	1507735	RYGGRØR RAK KJØREHÅNDTAK FOLDBARE XS KUSCHALL CO	1
10	SP1606526	(Backrest tube straight XXS for backrest foldable, pair)	1
11	SP1606527	(Backrest tube straight XS for backrest foldable, pair)	1
12	1432083	FESTEKIT FOR RYGGBÅND CHAMPION KUSCHALL	1
13	1525126	KJØREHÅNDTAK MINI SORT PLAST KUSCHALL	1
14	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1
15	1507733	KJØREHÅNDTAK FOLDBARE KUSCHALL CO PAR	1

(Push handles height adjustable)

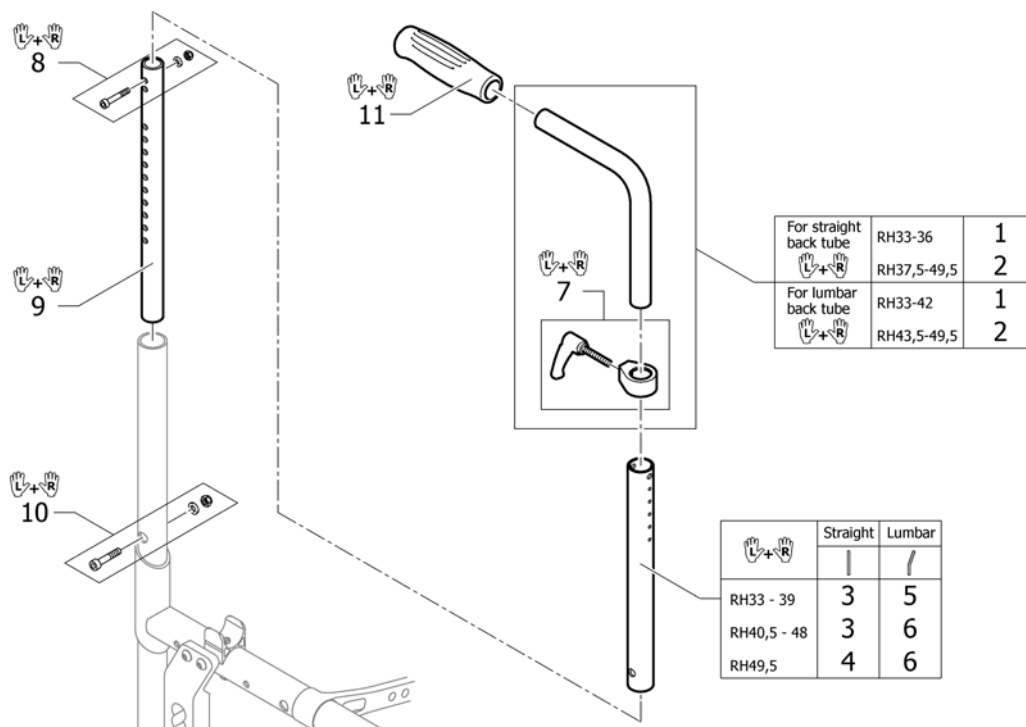


1	1438203	KJØREHÅNDTAK INTERGRERT KORT 20CM KÜSCHALL PAR	1
2	1438204	KJØREHÅNDT INTERGR.HØREG KÜSPAR	1
3	1507753	RYGGRØR L 280MM KUSCHALL PAR FOR 1438204	1
4	1509103	RYGGRØR FOR INTEGR. KJØREH VREG RH 39 PAR	1
5	1509117	(Backrest tube L.280mm lumbar to integrated push handles, pair)	1
6	1509122	(Backrest tube L.355mm lumbar to integrated push handles, pair)	1
7	1549985	JUSTERING HREG PAR KJØREHÅNDTAK KUSCHALL .	1
8	1540071	(Screw kit for backrest tube)	1
9	1507747	TELESKOP RYGGRØR PAR KÜS.PAR	1
10	1540088	FESTE KJØREHÅNDTAK LEDSAGERBREMS KUSCHALL	1
11	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1

(Push handles height adjustable for backrest angle adjustable)

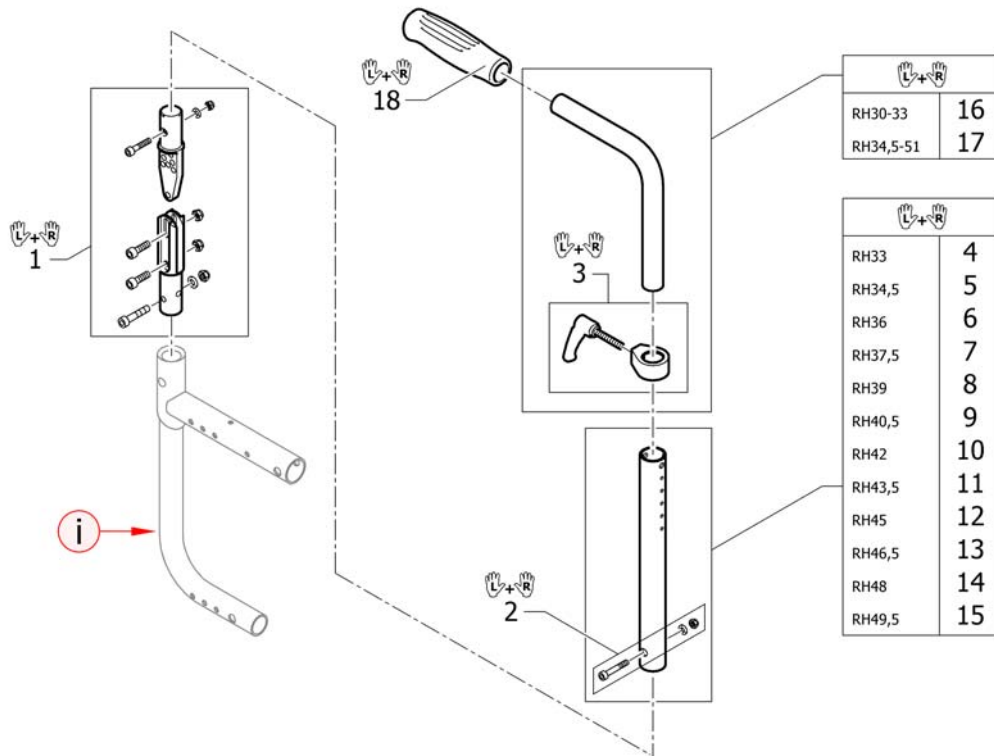


i	-	(A special rear frame is necessary. Please see chapter CHASSIS.)	0
1	1540094	RYGG VINKEL SETT	1
2	1540071	(Screw kit for backrest tube)	1
3	1549985	JUSTERING HREG PAR KJØREHÅNDTAK KUSCHALL .	1
4	1507753	RYGGRØR L 280MM KUSCHALL PAR FOR 1438204	1
5	1507862	RYGGRØR RAK 295MM FOR INT KJØRHÅNDT PAR	1
6	1508438	(Backrest tube L.310mm straight to integrated push handles, pair)	1
7	1508466	(Backrest tube L.325mm straight to integrated push handles, pair)	1
8	1508498	(Backrest tube L.340mm straight to integrated push handles, pair)	1
9	1509103	RYGGRØR FOR INTEGR. KJØREH VREG RH 39 PAR	1
10	1509104	RYGGRØR FOR INTEGR. KJØREH VREG RH 40,5 PAR	1
11	1509105	RYGGRØR FOR INTEGR. KJØREH VRG RYGGH 42 PAR	1
12	1509106	RYGGRØR FOR INTEGR. KJØREH VRG RYGGH 43,5PAR	1
13	1509107	(Backrest tube L.415mm straight to integrated push handles, pair)	1
14	1509108	RYGGRØR RH46,5 RETT FOR INTEGRERTE KJØREH PAR	1
15	1509109	(Backrest tube L.445mm straight to integrated push handles, pair)	1
16	1438203	KJØREHÅNDTAK INTERGRERT KORT 20CM KÜSCHALL PAR	1
17	1438204	KJØREHÅNDT INTERGR.HØREG KÜSPAR	1
18	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1

(Kuschall backrest parts for Matrix backs)

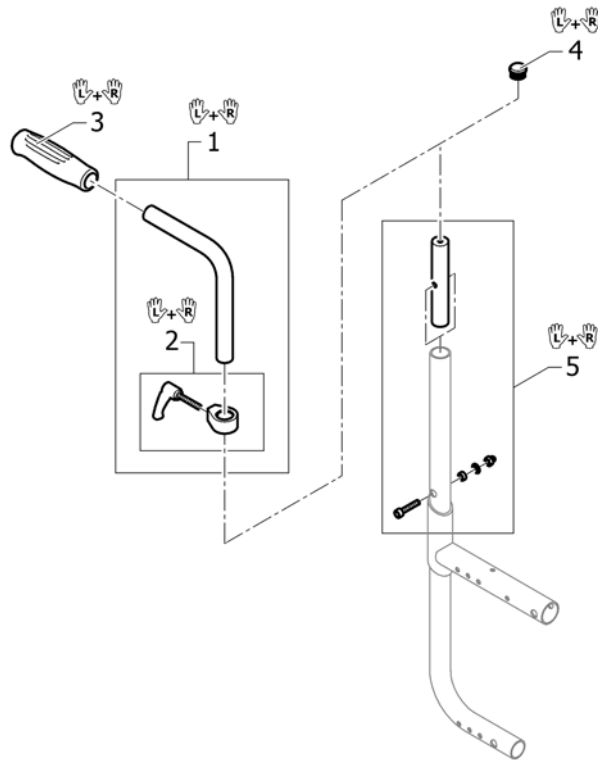


1	1438203	KJØREHÅNDTAK INTERGRERT KORT 20CM KÜSCHALL PAR	1
2	1549985	JUSTERING HREG PAR KJØREHÅNDTAK KUSCHALL .	1
3	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1
4	1561018	ENDEPLUGG RYGGRØR KUSCHALL Ø24MM	1
5	SP1602179	(Backrest bolt with fixation, pair)	1

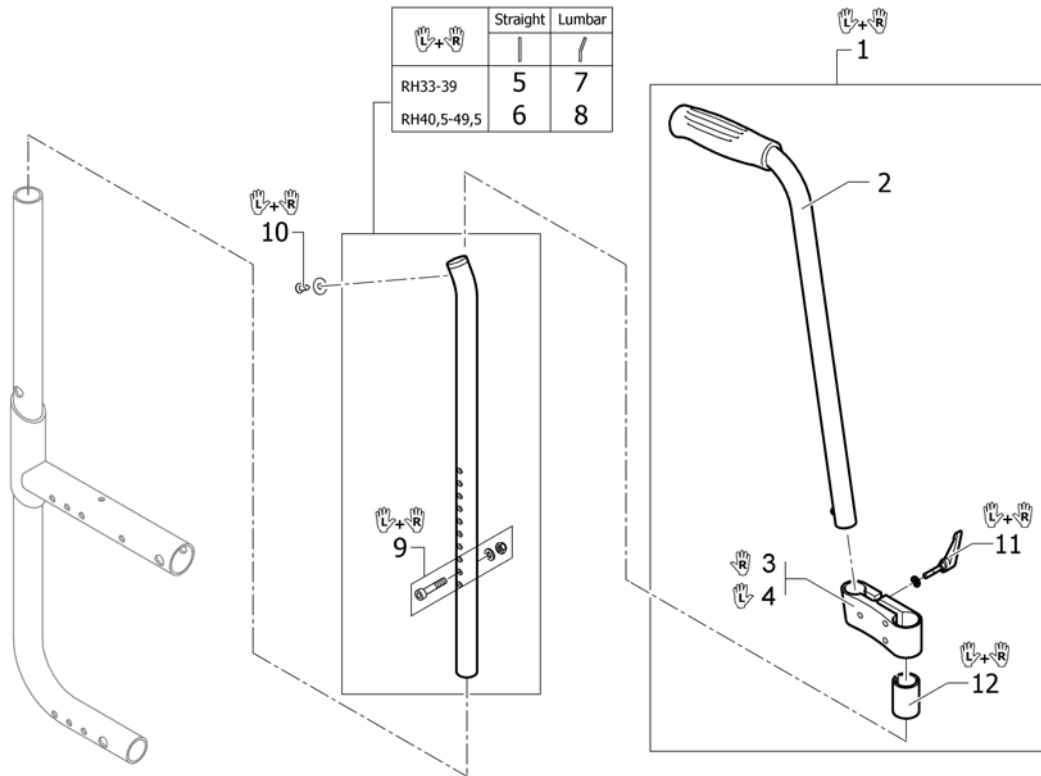


1	1517876	KJØREHÅNDTAK KOMPL PAR FESTE PÅ RYGGALLE	1
2	1438938	BAKENFORLIG KJØREH KPL UTEN FESTE KUSCHALL	2
3	1438939	FESTE BAKENFORLIG KJØREH HØYRE ALLE STOLER	1
4	1438940	FESTE BAKENFORLIG KJØREH VENSTRE ALLE STOLER	1
5	1540089	(Backrest tubes L without push handles, pair)	1
6	1540090	RYGGRØR U/KJØREHÅNDTAK MEDIUM LIFE KUSCHALL	1
7	1540091	(Backrest tubes lumbar L without push handles, pair)	1
8	1540092	(Backrest tubes lumbar XL without push handles, pair)	1
9	1540088	FESTE KJØREHÅNDTAK LEDSAGERBREMS KUSCHALL	1
10	1432083	FESTEKIT FOR RYGGBÅND CHAMPION KUSCHALL	1
11	1529799	PLASTHÅNDTAK FOR REG AV BEKENFORL KJØREHÅNDTAK	1
12	1581293	(Sleeves for mounting fixture, for rearsset push handles, pair)	1

(Backrest stabilization bar)

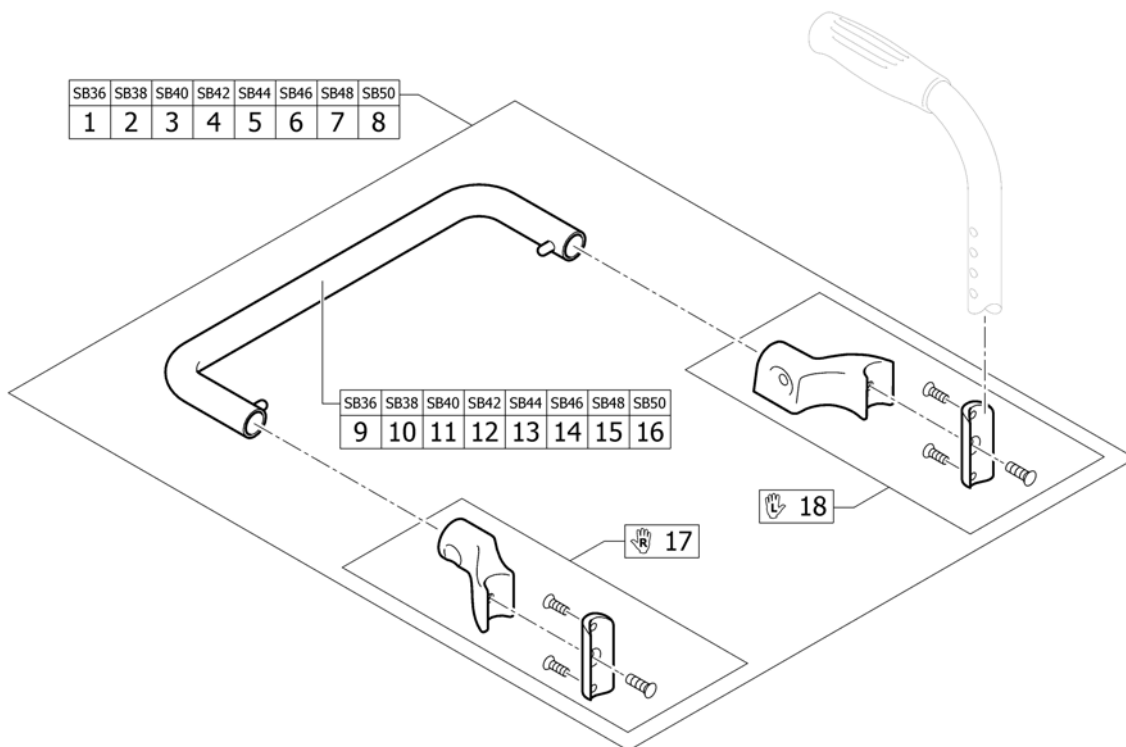


1	1517980	AVSTIVERBØYLE RYGG SB36 M/FESTE KUSCHALL	1
2	1517981	(Stabilizing bar complete SB38 for backrest tubes Ø20)	1
3	1517982	AVSTIVERBØYLE RYGG SB40T M/FESTE KUSCHALL COMPAKT	1
4	1517983	AVSTIVERBØYLE RYGG SB42T M/FESTE KUSCHALL COMPAKT	1
5	1517984	(Stabilizing bar complete SB44 for backrest tubes Ø20)	1
6	1517985	AVSTIVERBØYLE RYGG SB46T M/FESTE KUSCHALL COMPAKT	1
7	1517986	AVSTIVERBØYLE RYGG SB48 M/FESTE KUSCHALL	1
8	1517987	AVSTIVERBØYLE RYGG SB50 M/FESTE KUSCHALL	1
9	1518000	(Stabilizing bar SB36)	1
10	1518001	(Stabilizing bar SB38)	1
11	1518002	(Stabilizing bar SB40)	1
12	1518003	(Stabilizing bar SB42)	1
13	1518004	(Stabilizing bar SB44)	1
14	1518005	(Stabilizing bar SB46)	1
15	1518006	(Stabilizing bar SB48)	1
16	1518007	(Stabilizing bar SB50)	1
17	1518009	(Fixation kit to stabilizing bar, right, for backrest tubes Ø20)	1
18	1518008	(Fixation kit to stabilizing bar, left, for backrest tubes Ø20)	1

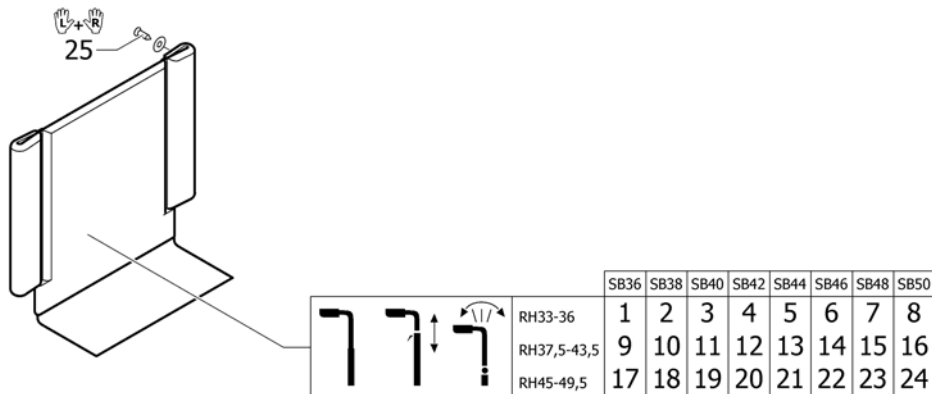


1	1552084	(Backrest cover SB36 medium)	1
2	1552085	(Backrest cover SB38 medium)	1
3	1552086	(Backrest cover SB40 medium)	1
4	1552087	(Backrest cover SB42 medium)	1
5	1552088	(Backrest cover SB44 medium)	1
6	1552089	(Backrest cover SB46 medium)	1
7	1552090	(Backrest cover SB48 medium)	1
8	1552091	(Backrest cover SB50 medium)	1
9	1552096	(Backrest cover SB36 long)	1
10	1552097	(Backrest cover SB38 long)	1
11	1552098	(Backrest cover SB40 long)	1
12	1552099	(Backrest cover SB42 long)	1
13	1552100	(Backrest cover SB44 long)	1
14	1552101	(Backrest cover SB46 long)	1
15	1552102	(Backrest cover SB48 long)	1
16	1552103	(Backrest cover SB50 long)	1
17	1552108	(Backrest cover SB36 X-long)	1
18	1552109	(Backrest cover SB38 X-long)	1
19	1552110	(Backrest cover SB40 X-long)	1
20	1552111	(Backrest cover SB42 X-long)	1
21	1552112	(Backrest cover SB44 X-long)	1
22	1552113	(Backrest cover SB46 X-long)	1
23	1552114	(Backrest cover SB48 X-long)	1
24	1552115	(Backrest cover SB50 X-long)	1
25	1432083	FESTEKIT FOR RYGGBÅND CHAMPION KÜSCHALL	1


(Bands to adjustable backrest)

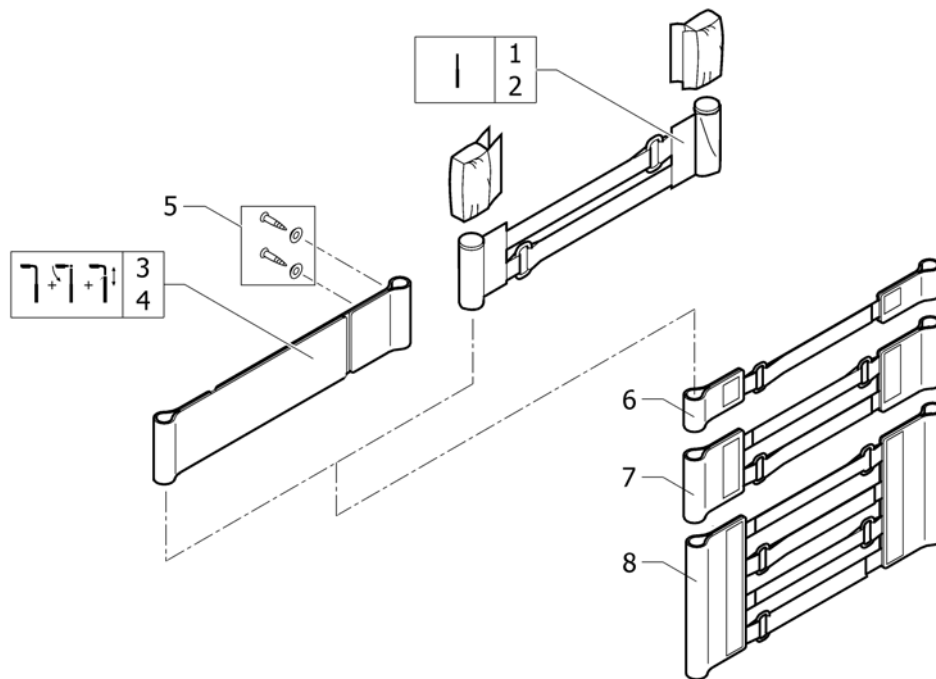

1	1513172	VELCROBÄND ENDESTYKKE 5 CM	1
2	1513173	VELCROBÄND ENDESTYKKE 10 CM	1
3	1513170	VELCROREM 2-DELT 5 CM PAR	1
4	1513171	VELCROBÄND TOPP 10 CM 2-DELT ALLE 2008->	1
5	1432083	FESTEKIT FOR RYGGBÄND CHAMPION KÜSCHALL	1
6	1513167	VELCROBÄND JUSTERBART M 1 STROPP K4 2008-MOD	1
7	1513168	VELCROBÄND JUSTERBART M 2 STROPP COMPACT-2004M	1
8	1513169	VELCROBÄND JUSTERBART M 4 STROPP KUSCHALL	1

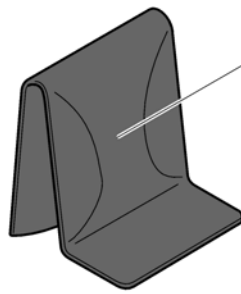


Std. (black)	SB36-38	SB40-42	SB44-46	SB48
RH33-36	1	2	3	4
RH37,5-42	5	6	7	8
RH43,5-49,5	9	10	11	12

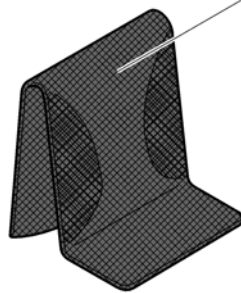

AeroBack	SB36-38	SB40-42	SB44-46	SB48
RH33-36	13	14	15	16
RH37,5-42	17	18	19	20
RH43,5-49,5	21	22	23	24

1	1583109	(Cover black to adjustable backrest, SB36-38, short)	1
2	1583110	(Cover black to adjustable backrest, SB40-42, short)	1
3	1583111	(Cover black to adjustable backrest, SB44-46, short)	1
4	1583112	(Cover black to adjustable backrest, SB48-50, short)	1
5	1583113	(Cover black to adjustable backrest, SB36-38, medium)	1
6	1583114	(Cover black to adjustable backrest, SB40-42, medium)	1
7	1583115	(Cover black to adjustable backrest, SB44-46, medium)	1
8	1583116	(Cover black to adjustable backrest, SB48-50, medium)	1
9	1583117	(Cover black to adjustable backrest, SB36-38, long)	1
10	1583118	(Cover black to adjustable backrest, SB40-42, long)	1
11	1583119	(Cover black to adjustable backrest, SB44-46, long)	1
12	1583120	(Cover black to adjustable backrest, SB48-50, long)	1
13	1512157	(Cover 'Aero' to adjustable backrest, SB36-38, short)	1
14	1512158	RYGGTREKK SB40-42 KORT AEROBACK	1
15	1512159	(Cover 'Aero' to adjustable backrest, SB44-46, short)	1
16	1512160	(Cover 'Aero' to adjustable backrest, SB48-50, short)	1
17	1512162	RYGGTREKK SB36-38 MEDIUM AEROBACK KUSCHALL	1
18	1512163	RYGGTREKK SB40-42 MEDIUM AEROBACK KUSCHALL	1
19	1512164	RYGGTREKK SB44-46 MEDIUM AEROBACK KUSCHALL	1
20	1512165	RYGGTREKK SB48-50 MEDIUM AEROBACK KUSCHALL	1
21	1512167	RYGGTREKK AEROBACK SB36-38 LANG VELCRO RYGG	1
22	1512168	RYGGTREKK SB40-42 LANG AEROBACK CHAMPION-2008	1
23	1512169	RYGGTREKK SB44-46 LANG AEROBACK KUSCHALL	1
24	1512170	(Cover 'Aero' to adjustable backrest, SB48-50, long)	1


(LEGRESTS)

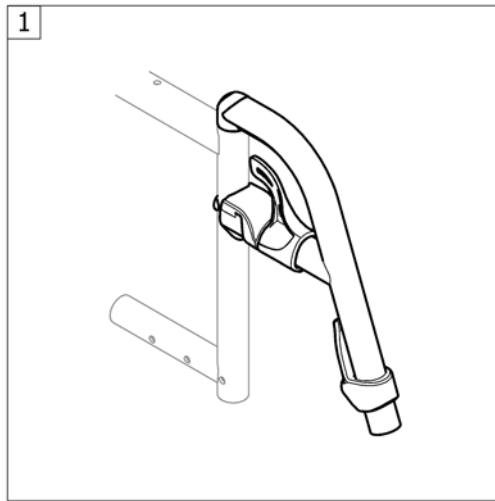

1 -




(Legrest hanger)

1

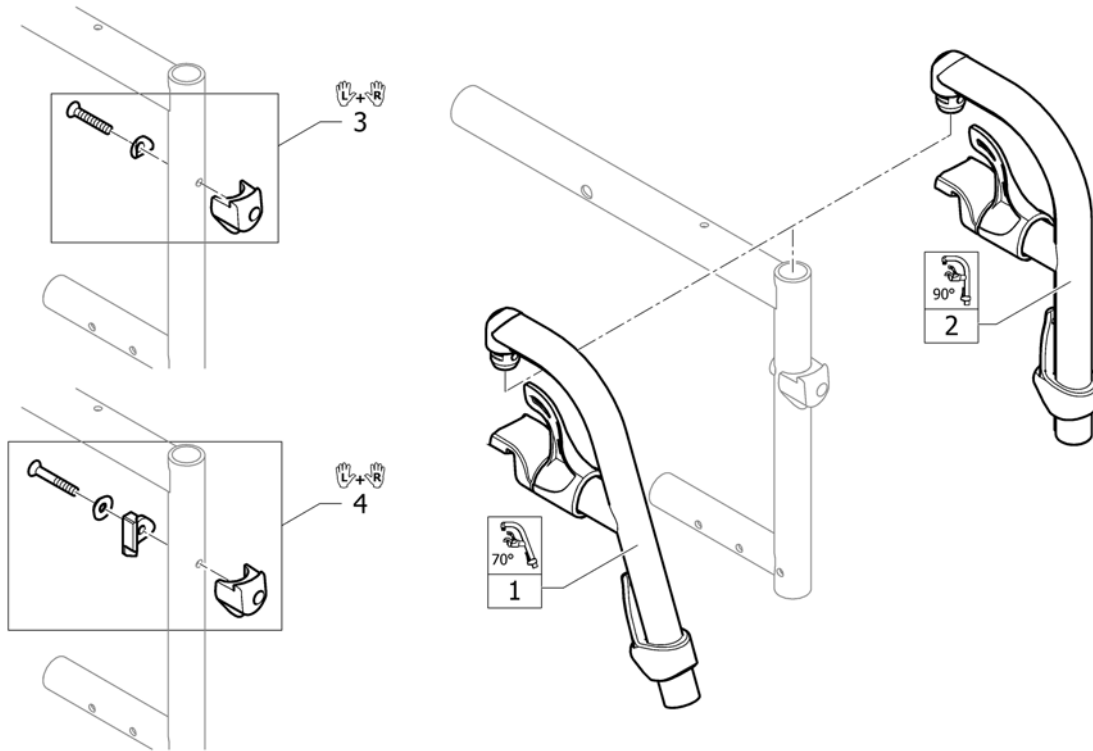


1	SP1584369-0453	(Swing Away 80°, attract blue, complete)	2
1	SP1584369-0454	(Swing Away 80°, attract black, complete)	2
1	SP1584369-0455	(Swing Away 80°, attract grey, complete)	2
1	SP1584369-0456	(Swing Away 80°, attract red, complete)	2
2	SP1584370-0453	(Swing Away 90°, attract blue, complete)	2
2	SP1584370-0454	(Swing Away 90°, attract black, complete)	2
2	SP1584370-0455	(Swing Away 90°, attract grey, complete)	2
2	SP1584370-0456	(Swing Away 90°, attract red, complete)	2
3	SP1603392	(Position mechanism for swing-away, kit, pair)	1
4	SP1603777	(Rotating limiter for swing-away, kit, pair)	1

(FOOTREST)

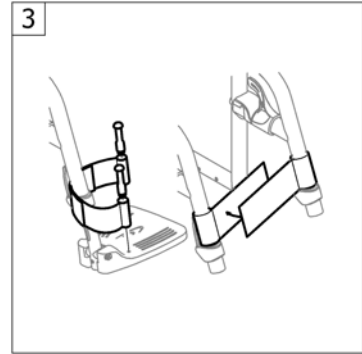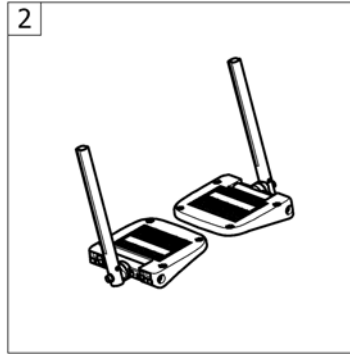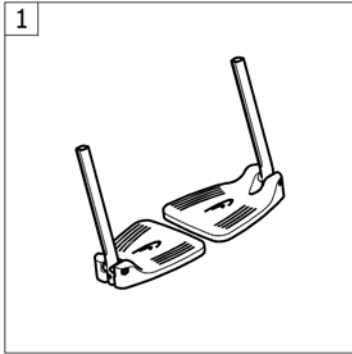

1	-	(Footrest 2-piece)	1
2	-	(Footrest 2-piece, angle adjustable)	1
3	-	(Footbands)	1

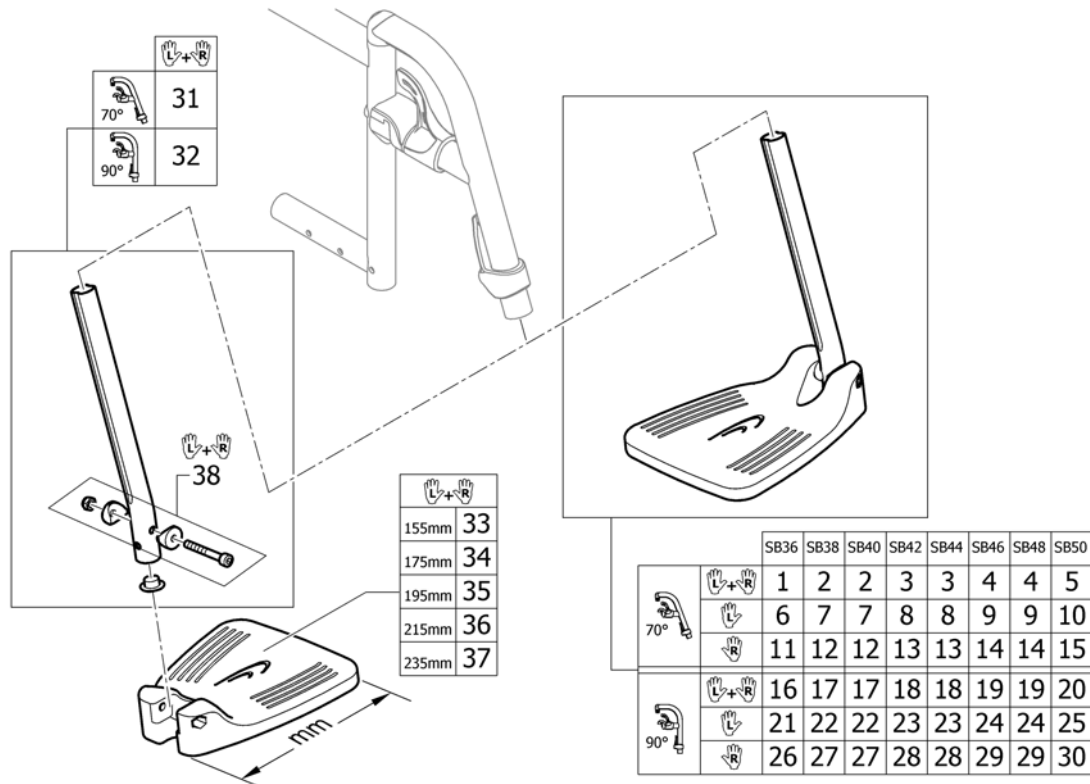


1	SP1603867	(Footrest 2-pc. 155mm, for frame 70°, pair, complete)	1
2	SP1603868	(Footrest 2-pc. 175mm, for frame 70°, pair, complete)	1
3	SP1603869	(Footrest 2-pc. 195mm, for frame 70°, pair, complete)	1
4	SP1603884	(Footrest 2-pc. 215mm, for frame 70°, pair, complete)	1
5	SP1603885	(Footrest 2-pc. 235mm, for frame 70°, pair, complete)	1
6	SP1603886	(Footrest 2-pc. 155mm, for frame 70°, left, complete)	1
7	SP1603887	(Footrest 2-pc. 175mm, for frame 70°, left, complete)	1
8	SP1603888	(Footrest 2-pc. 195mm, for frame 70°, left, complete)	1
9	SP1603889	(Footrest 2-pc. 215mm, for frame 70°, left, complete)	1
10	SP1603890	(Footrest 2-pc. 235mm, for frame 70°, left, complete)	1
11	SP1603891	(Footrest 2-pc. 155mm, for frame 70°, right, complete)	1
12	SP1603892	(Footrest 2-pc. 175mm, for frame 70°, right, complete)	1
13	SP1603893	(Footrest 2-pc. 195mm, for frame 70°, right, complete)	1
14	SP1603894	(Footrest 2-pc. 215mm, for frame 70°, right, complete)	1
15	SP1603895	(Footrest 2-pc. 235mm, for frame 70°, right, complete)	1
16	SP1603908	(Footrest 2-pc. 155mm, for frame 90°, pair, complete)	1
17	SP1603909	(Footrest 2-pc. 175mm, for frame 90°, pair, complete)	1
18	SP1603910	(Footrest 2-pc. 195mm, for frame 90°, pair, complete)	1
19	SP1603911	(Footrest 2-pc. 215mm, for frame 90°, pair, complete)	1
20	SP1603912	(Footrest 2-pc. 235mm, for frame 90°, pair, complete)	1
21	SP1603913	(Footrest 2-pc. 155mm, for frame 90°, left, complete)	1
22	SP1603914	(Footrest 2-pc. 175mm, for frame 90°, left, complete)	1
23	SP1603915	(Footrest 2-pc. 195mm, for frame 90°, left, complete)	1
24	SP1603940	(Footrest 2-pc. 215mm, for frame 90°, left, complete)	1
25	SP1603941	(Footrest 2-pc. 235mm, for frame 90°, left, complete)	1
26	SP1603942	(Footrest 2-pc. 155mm, for frame 90°, right, complete)	1
27	SP1604002	(Footrest 2-pc. 175mm, for frame 90°, right, complete)	1
28	SP1604003	(Footrest 2-pc. 195mm, for frame 90°, right, complete)	1
29	SP1604004	(Footrest 2-pc. 215mm, for frame 90°, right, complete)	1
30	SP1604005	(Footrest 2-pc. 235mm, for frame 90°, right, complete)	1
31	SP1603907	(Telescopic tube kit for footrest 2-pc. for frame 70°, pair)	1

(Footrest 2-piece)


32	SP1604006	(Telescopic tube kit for footrest 2-pc. for frame 90°, pair)	1
33	1579331	(Footrest 2-pc. 155mm, pair)	1
34	1579332	(Footrest 2-pc. 175mm, pair)	1
35	1579333	(Footrest 2-pc. 195mm, pair)	1
36	1579334	(Footrest 2-pc. 215mm, pair)	1
37	1579335	(Footrest 2-pc. 235mm, pair)	1
38	1582352	(Fixation kit for telescopic tube on footrest 2-pcs., pair)	1

(Footrest 2-piece, angle adjustable)

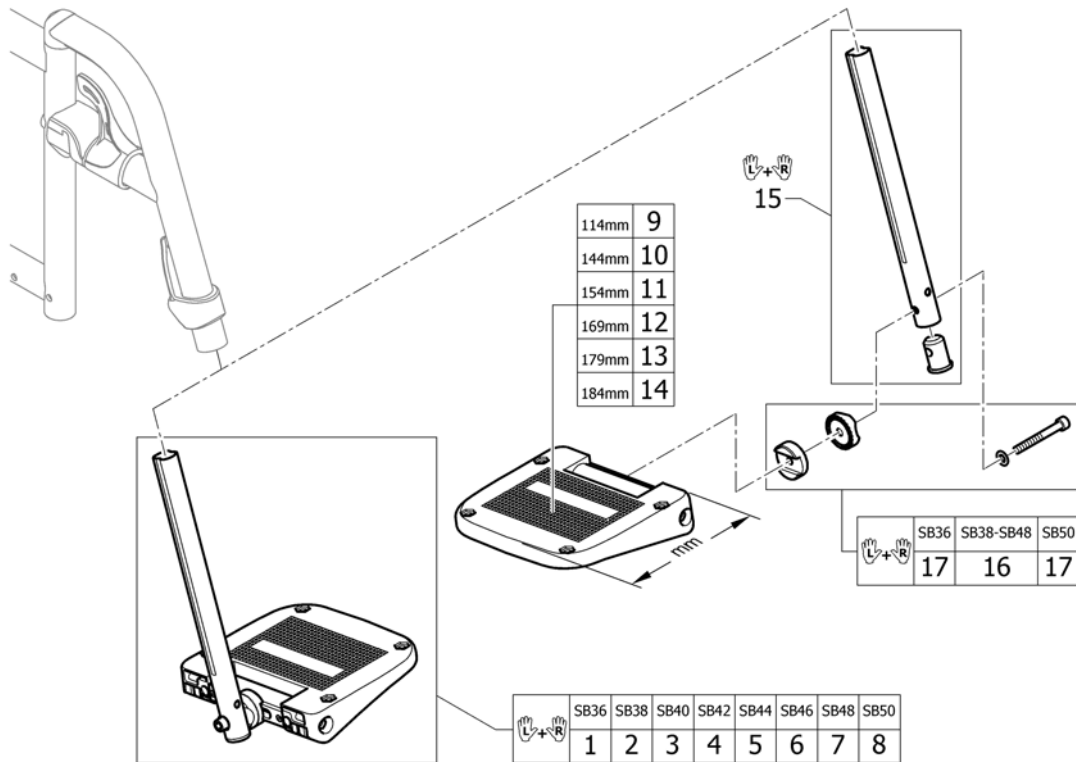


1	SP1603789	(Footrest 2-pc. angle adjustable SB36, pair, complete)	1
2	SP1603790	(Footrest 2-pc. angle adjustable SB38, pair, complete)	1
3	SP1603791	(Footrest 2-pc. angle adjustable SB40, pair, complete)	1
4	SP1603792	(Footrest 2-pc. angle adjustable SB42, pair, complete)	1
5	SP1603793	(Footrest 2-pc. angle adjustable SB44, pair, complete)	1
6	SP1603835	(Footrest 2-pc. angle adjustable SB46, pair, complete)	1
7	SP1603836	(Footrest 2-pc. angle adjustable SB48, pair, complete)	1
8	SP1603837	(Footrest 2-pc. angle adjustable SB50, pair, complete)	1
9	SP1603838	(Footrest 2-pc. angle adjustable 114mm)	2
10	SP1603839	(Footrest 2-pc. angle adjustable 144mm)	2
11	SP1603841	(Footrest 2-pc. angle adjustable 154mm)	2
12	SP1603842	(Footrest 2-pc. angle adjustable 169mm)	2
13	SP1603843	(Footrest 2-pc. angle adjustable 179mm)	2
14	SP1603844	(Footrest 2-pc. angle adjustable 184mm)	2
15	SP1603846	(Telescopic tube kit for footrest 2-pc. angle adjustable, pair)	1
16	SP1603865	(Lateral fixation, short, pair)	1
17	SP1603866	(Lateral fixation, long, pair)	1

(Footbands)

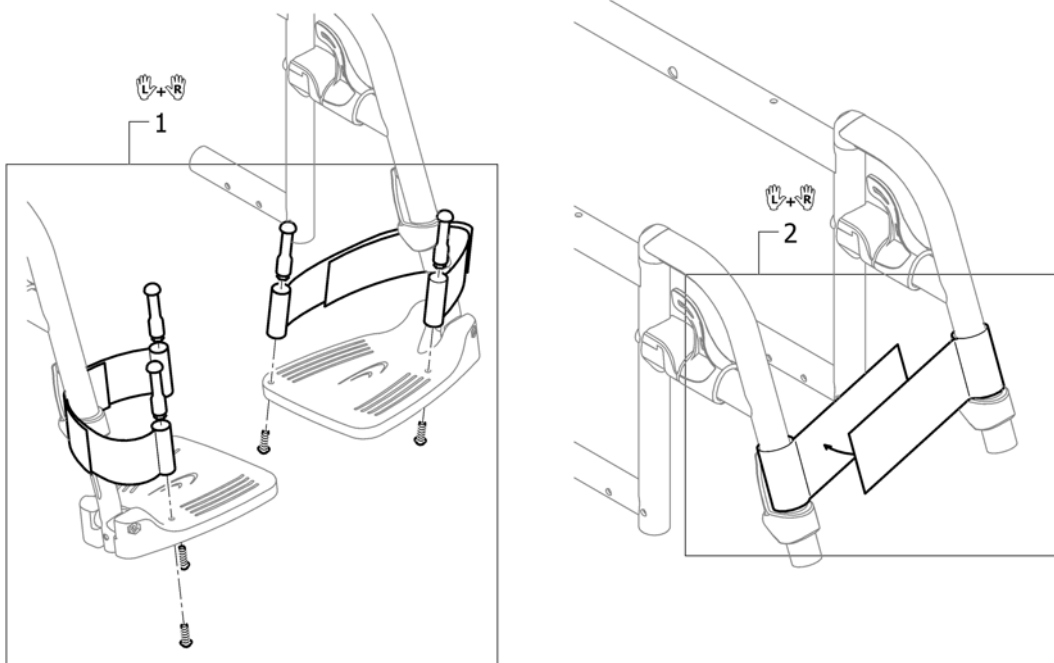


1	1437834	HÆLBÅND PAR. M/SKRUER KUSCHALL LIFE	1
2	1536082	LEGGBÅND SB40 COMPACT 09	1

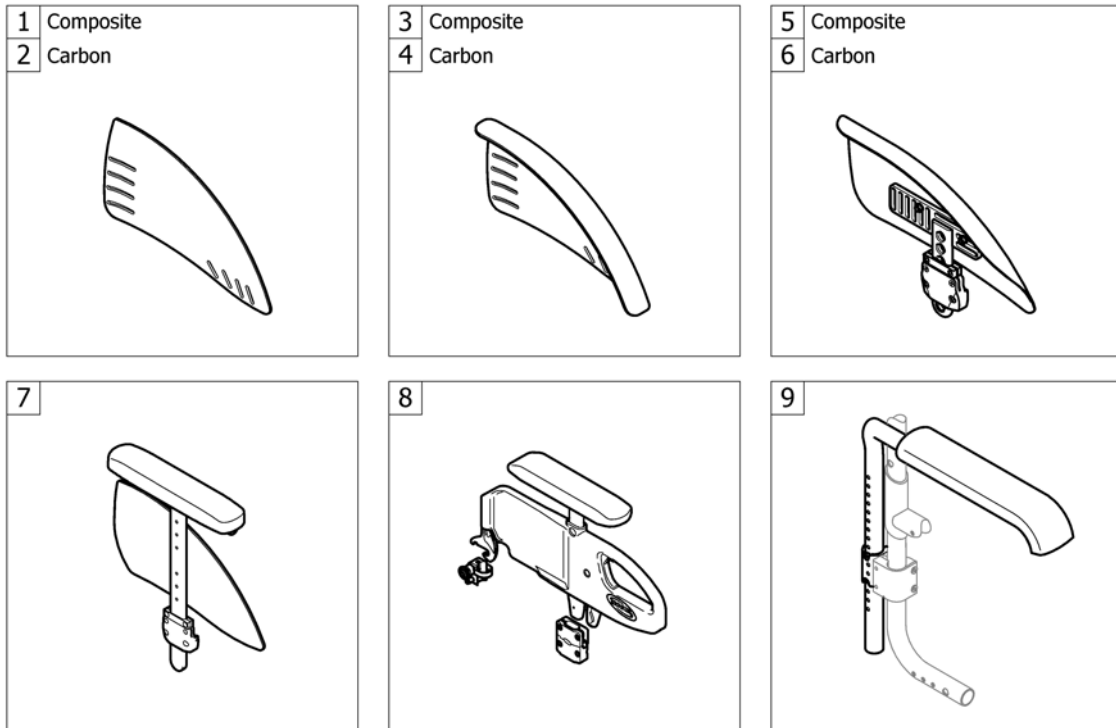


1	-	(Clothes-guards composite)	1
2	-	(Clothes-guards carbon)	1
3	-	(Mudguards composite)	1
4	-	(Mudguards carbon)	1
5	-	(Mudguards composite removable)	1
6	-	(Mudguards carbon removable)	1
7	-	(Siderest height adjustable)	1
8	-	(Siderest with desk function)	1
9	-	(Tubular armrest)	1

01.07.2016 →

(Clothes-guards composite)

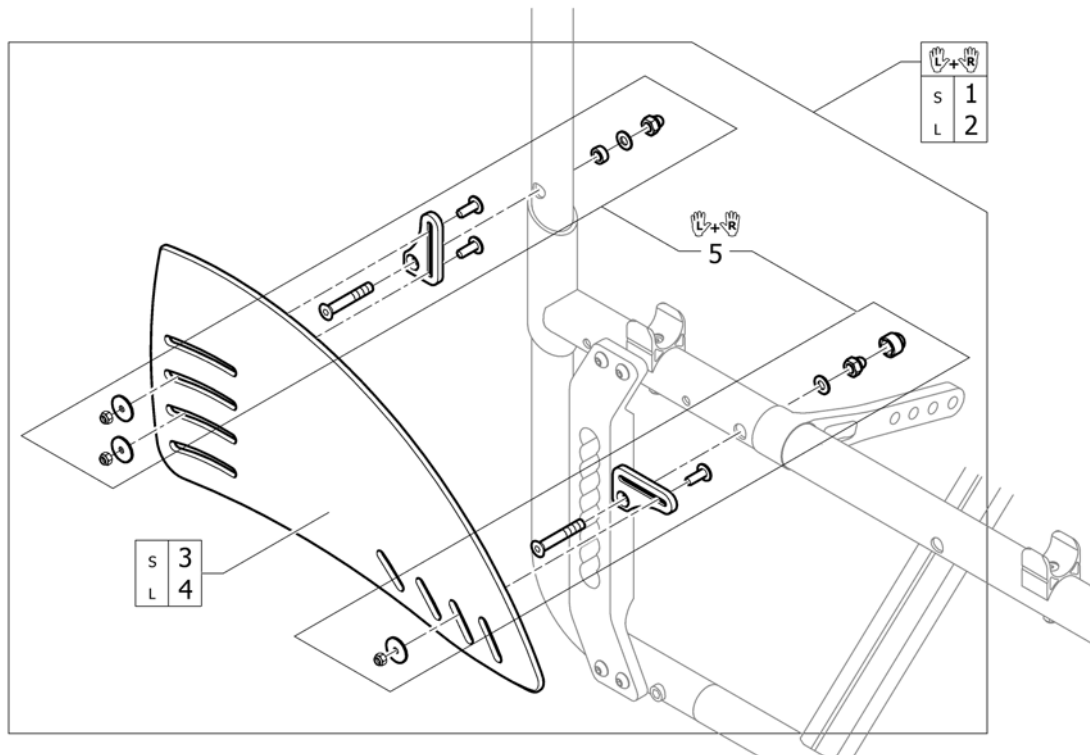


1	SP1604011	(Clothes-guards composite S with fixation kit, pair)	1
2	SP1604012	(Clothes-guards composite L with fixation kit, pair)	1
3	SP1604013	(Clothes-guard composite S)	2
4	SP1604029	(Clothes-guard composite L)	2
5	SP1604771	(Fixation kit for clothes-guard and mudguard composite)	1

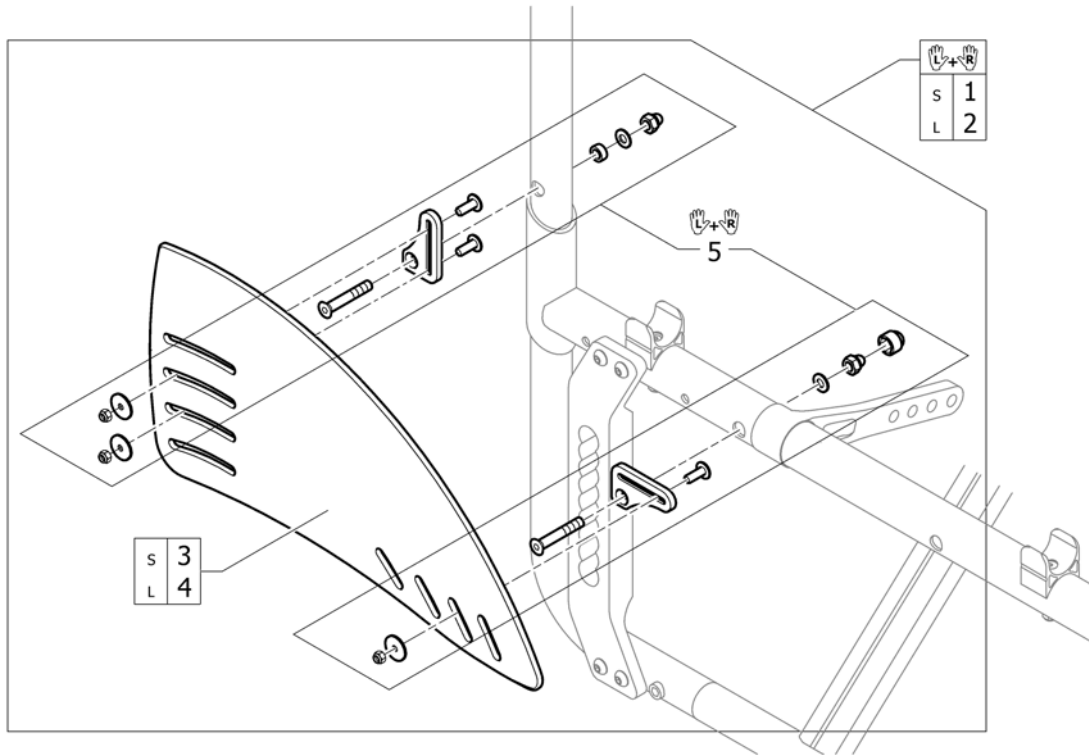


1	SP1604880	(Clothes-guards carbon S with fixation kit, pair)	1
2	SP1604945	(Clothes-guards carbon L with fixation kit, pair)	1
3	1540003	KLESBESKYTTER LITEN	2
4	1540004	KLESBESKYTTER STOR	2
5	SP1604795	(Fixation kit for clothes-guard and mudguard carbon)	1

(Mudguards composite)

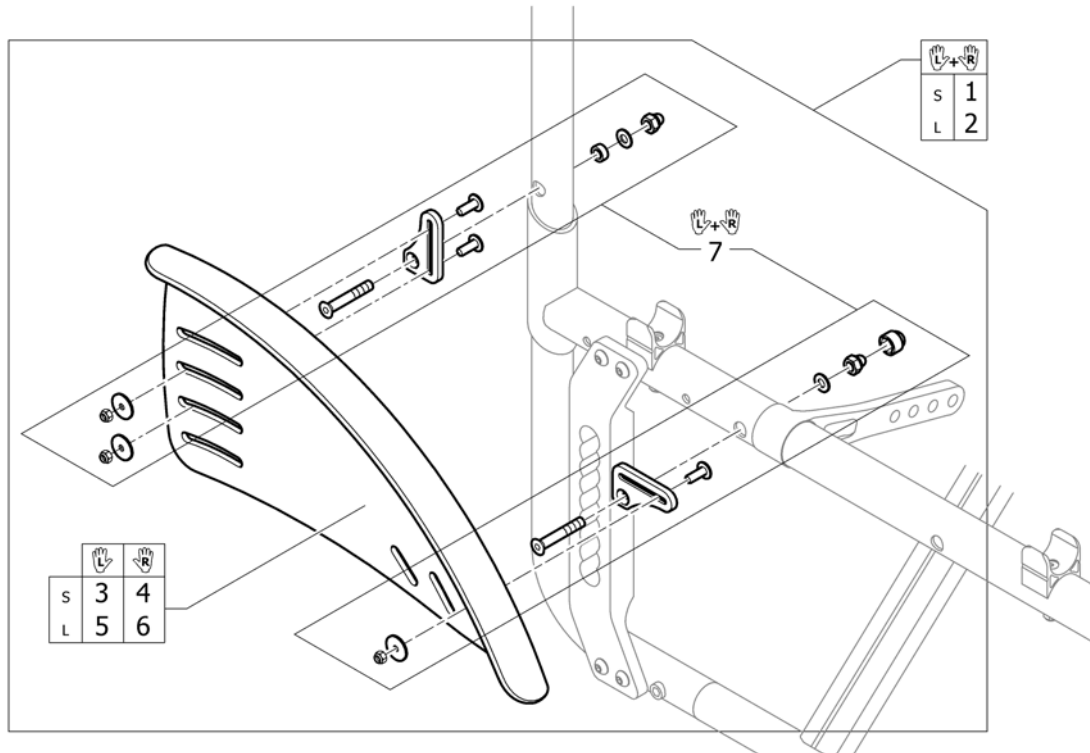


1	SP1604772	(Mudguard composite S with fixation kit, pair)	1
2	SP1604773	(Mudguard composite L with fixation kit, pair)	1
3	SP1604774	(Mudguard composite S left)	1
4	SP1604775	(Mudguard composite S right)	1
5	SP1604776	(Mudguard composite L left)	1
6	SP1604777	(Mudguard composite L right)	1
7	SP1604771	(Fixation kit for clothes-guard and mudguard composite)	1

(Mudguards carbon)

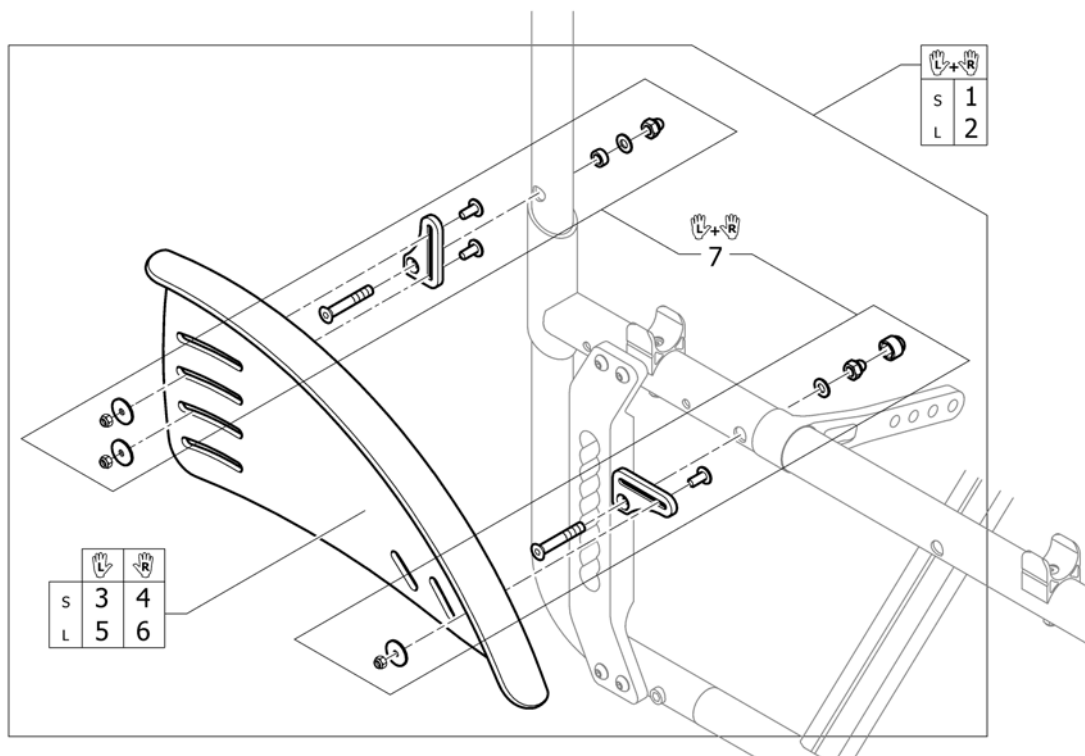


1	SP1604793	(Mudguard carbon S with fixation kit, pair)	1
2	SP1604794	(Mudguard carbon L with fixation kit, pair)	1
3	1529781	HJULSKJERM FAST KARBON S HORT VE UL CO	1
4	1529782	HJULSKJERM FAST KARBON S HORT HØ UL CO	1
5	1529783	(Mudguard Carbon L long left)	1
6	1529784	(Mudguard Carbon L long right)	1
7	SP1604795	(Fixation kit for clothes-guard and mudguard carbon)	1

(Mudguards composite removable)

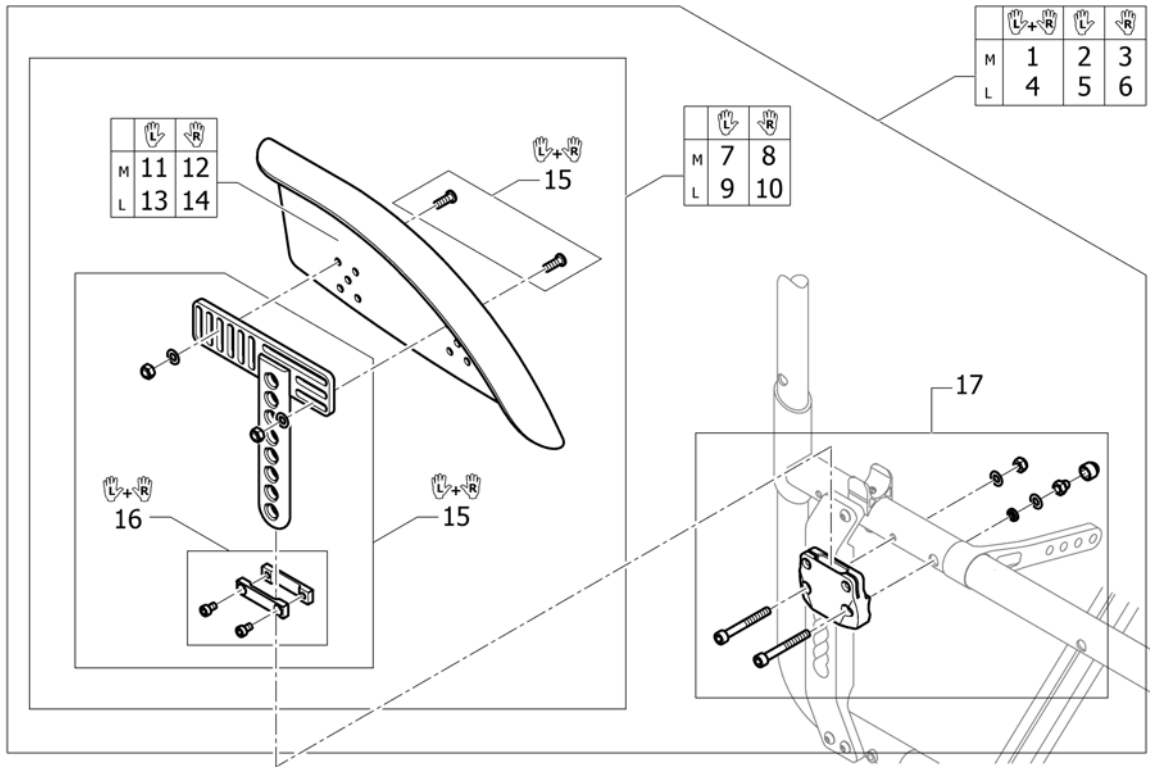


1	SP1604952	(Mudguard composite M long complete, pair)	1
2	SP1604953	(Mudguard composite left M long complete)	1
3	SP1604954	(Mudguard composite right M long complete)	1
4	SP1604955	(Mudguard composite L long complete, pair)	1
5	SP1604956	(Mudguard composite left L long complete)	1
6	SP1604957	(Mudguard composite right L long complete)	1
7	SP1604969	(Mudguard composite left M, holder long)	1
8	SP1604970	(Mudguard composite right M, holder long)	1
9	SP1604971	(Mudguard composite left L, holder long)	1
10	SP1604972	(Mudguard composite right L, holder long)	1
11	SP1604959	(Mudguard composite M left)	1
12	SP1604960	(Mudguard composite M right)	1
13	SP1604961	(Mudguard composite L left)	1
14	SP1604962	(Mudguard composite L right)	1
15	SP1604963	(Mudguard removable holder long, pair)	1
16	1547537	(Siderests height clamp kit, pair)	1
17	SP1604958	(Sidepart Adaption complete)	2

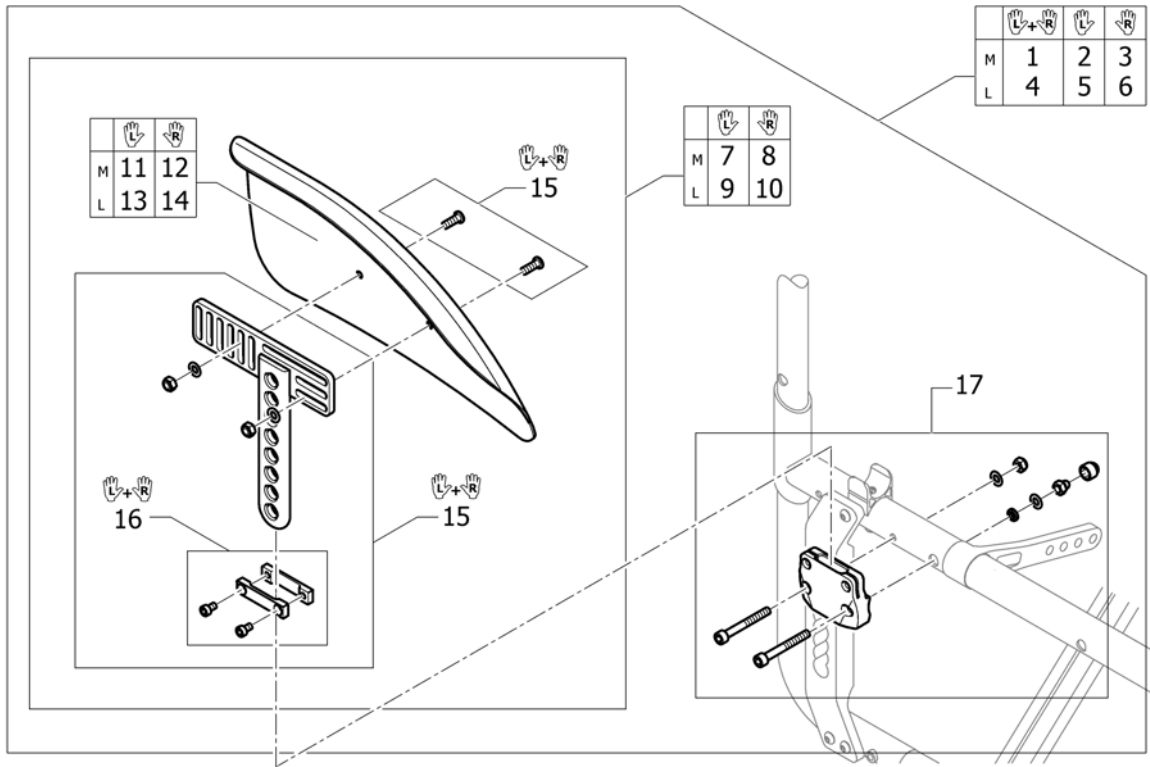


1	SP1604946	(Mudguard carbon M long complete, pair)	1
2	SP1604947	(Mudguard carbon left M long complete)	1
3	SP1604948	(Mudguard carbon right M long complete)	1
4	SP1604949	(Mudguard carbon L long complete, pair)	1
5	SP1604950	(Mudguard carbon left L long complete)	1
6	SP1604951	(Mudguard carbon right L long complete)	1
7	SP1604965	(Mudguard carbon left M, holder long)	1
8	SP1604966	(Mudguard carbon right M, holder long)	1
9	SP1604967	(Mudguard carbon left L, holder long)	1
10	SP1604968	(Mudguard carbon right L, holder long)	1
11	1510275	HJULSKJERM KARBON HØ MEDIUM KUSCHALL	1
12	1510276	HJULSKJERM KARBON VE MEDIUM KUSCHALL	1
13	1510277	HJULSKJERM KARBON VE LARGE KUSCHALL	1
14	1510278	HJULSKJERM KARBON HØ LARGE KUSCHALL	1
15	SP1604963	(Mudguard removable holder long, pair)	1
16	1547537	(Siderests height clamp kit, pair)	1
17	SP1604958	(Sidepart Adaption complete)	2

(Siderest height adjustable)

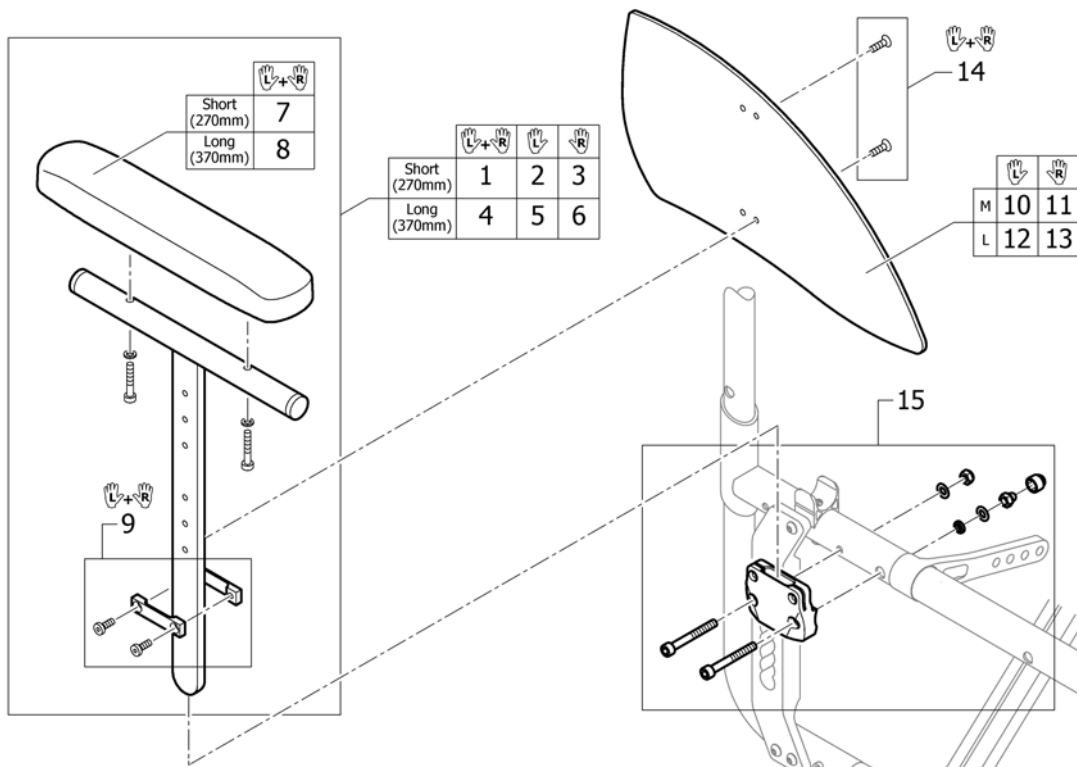


1	1517682	ARMLENE REG KORT PAR	1
2	1517678	ARMLEN KORT HREG VE CO UL	1
3	1517677	ARMLEN KORT HREG HØ CO UL 1.4.8->	1
4	1517683	ARMLEN LONG HREG PAR	1
5	1517680	ARMLEN HREG VE LANGT POLSTER CO UL FU 1.4.8->	1
6	1517679	ARMLEN HREG HØ LANGT POLSTER CO UL FU 1.4.8->	1
7	1439497	ARMLENS POLSTER KORT 27 C CO UL 2004 -> PAR	1
8	1439498	ARMLENS POLSTER LANG CO UL 2004-> PAR 36CM.	1
9	1547537	(Siderests height clamp kit, pair)	1
10	1517689	KLESBESKYTTER S VE FOR ARMLEN CO UL 2008->	1
11	1517690	KLESBESKYTTER S HØ FOR ARMLEN CO UL 2008->	1
12	1517691	KLESBESKYTTER L VE FOR ARMLEN CO UL 2008->	1
13	1517692	KLESBESKYTTER L HØ FOR ARMLEN CO UL 2008->	1
14	1517693	FESTE KLESBESKYTTER FOR ARMLEN CO UL 2008->	1
15	SP1604958	(Sidepart Adaption complete)	2


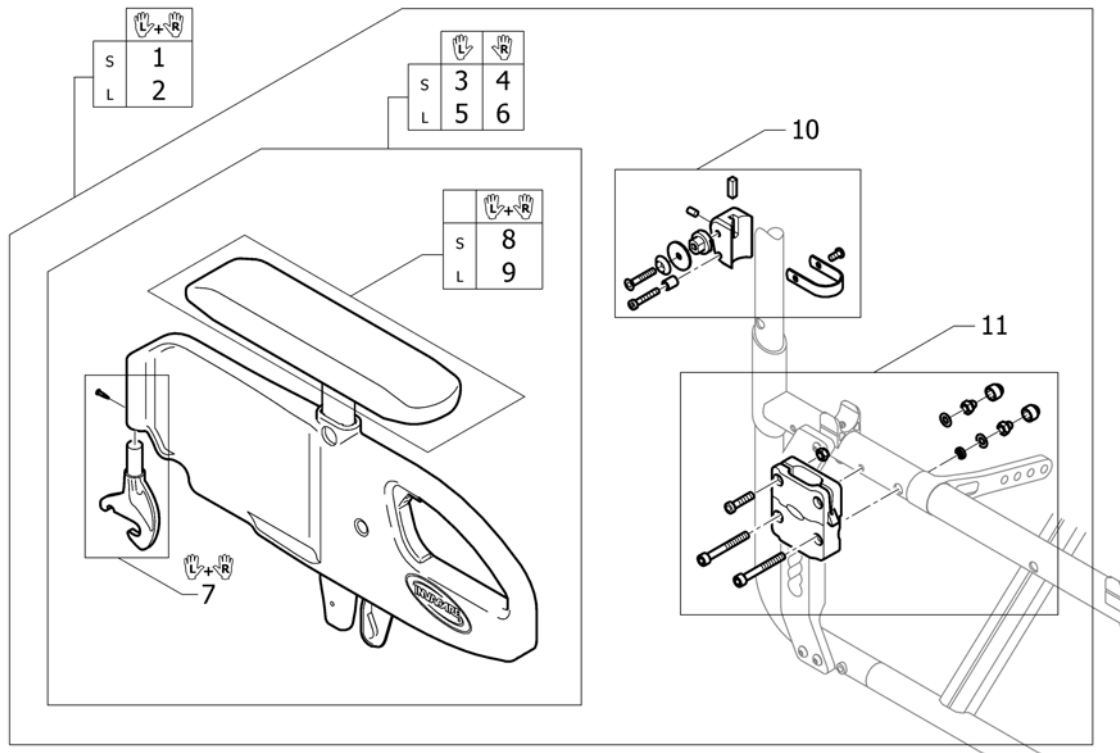


1	1540043	(Siderest with desk function, flip-back, height adj., removable, pair short, complete)	1
2	1540044	ARMLÉN DESK OPPFELLBAR HREG KPL MED FESTE PAR	1
3	1540040	(Siderest with desk function, flip-back, height adj., removable, short, left)	1
4	1540039	(Siderest with desk function, flip-back, height adj., removable, short, right)	1
5	1540042	(Siderest with desk function, flip-back, height adj., removable, long left)	1
6	1540041	(Siderest with desk function, flip-back, height adj., removable, long, right)	1
7	1540031	(Hook bracket for siderest, pair)	1
8	1521532	(Armpad short incl. Screws, pair)	1
9	1521533	(Armpad long incl. Screws, pair)	1
10	1540032	(Siderest with desk function fixation kit)	2
11	SP1604973	(Fixation kit front for siderest)	2

(Tubular armrest)

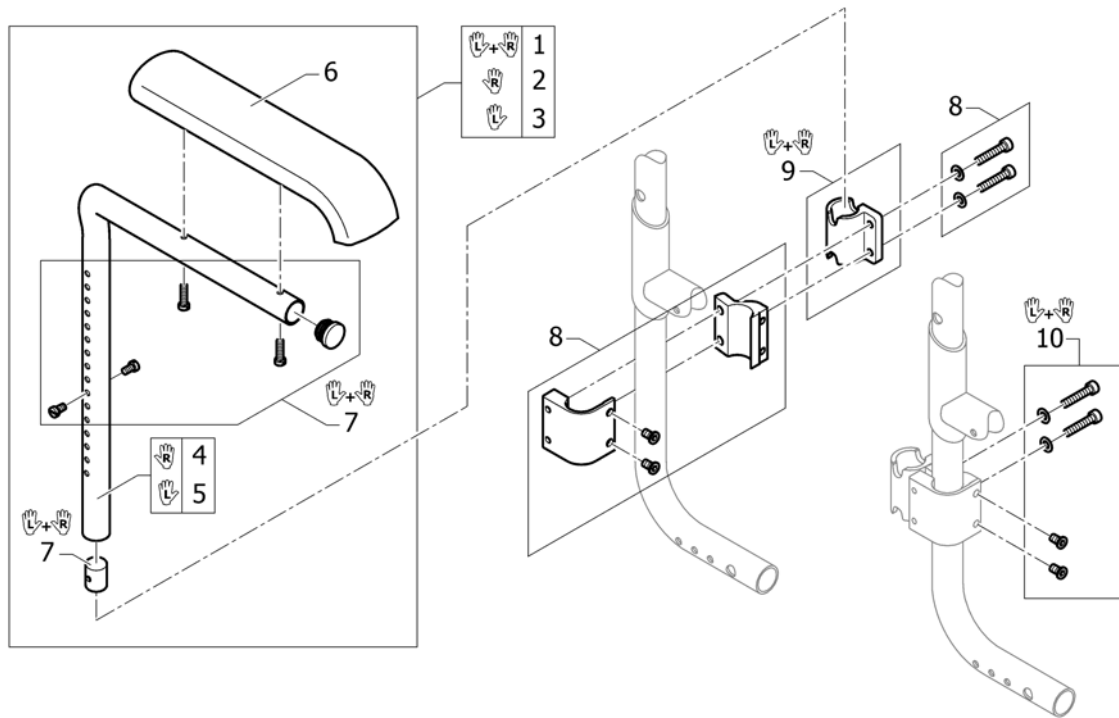


1	SP1606428	(Pair of armrests with standard pads, bent)	1
2	SP1606429	(Armrest with standard pad, bent, right)	1
3	SP1606430	(Armrest with standard pad, bent, left)	1
4	SP1606431	(Armrest right)	1
5	SP1606432	(Armrest left)	1
6	SP1589276	(Standard pad)	2
7	SP1606433	(Screws kit for tubular armrest, pair)	1
8	SP1604985	(Clamping piece to adaption)	2
9	SP1606434	(Armrest holder, pair)	1
10	SP1606435	(Fixation kit for clamping piece to adaption, pair)	1

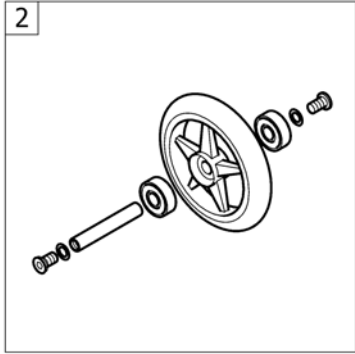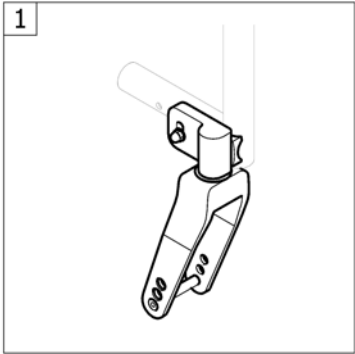



1	-	(Front wheel supporter)	1
2	-	(Front wheels)	1

(Front wheel supporter)

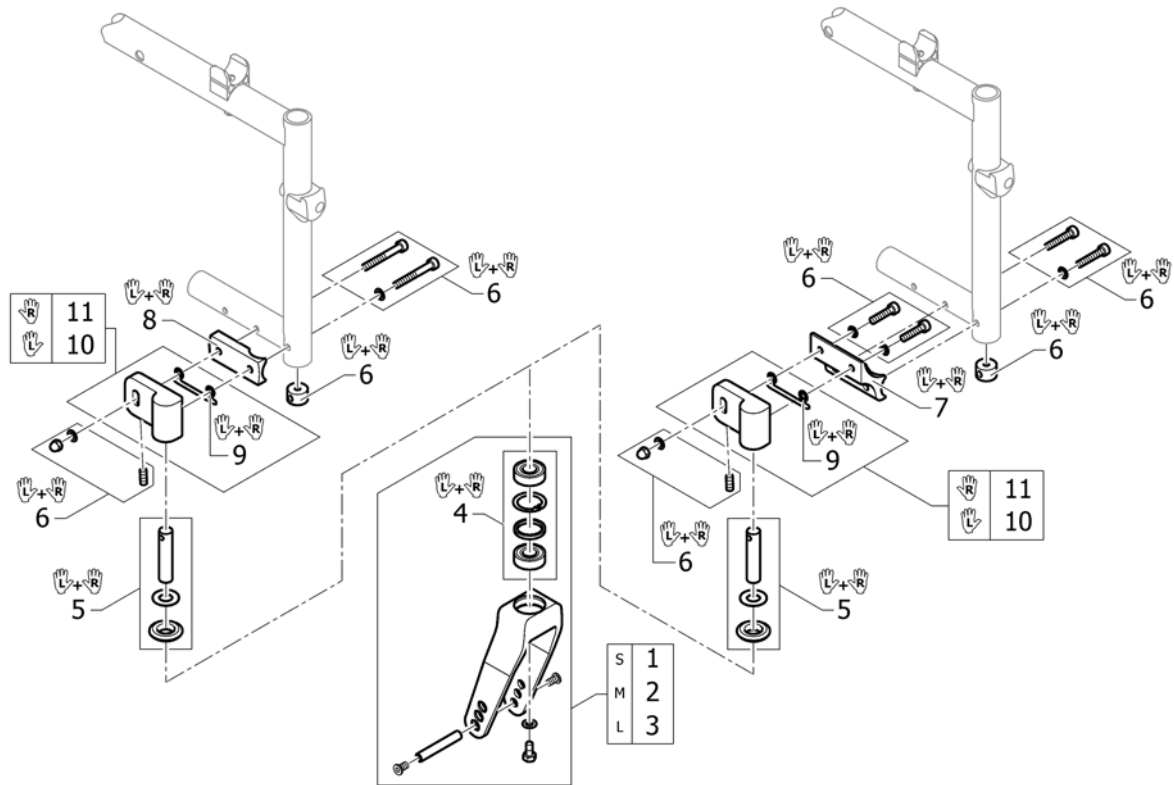


1	SP1603779-0004	(Front fork S, black anodized)	2
2	SP1603780-0004	(Front fork M, black anodized)	2
3	SP1603781-0004	(Front fork L, black anodized)	2
4	1510219	KULELAGER KIT	1
5	SP1603786	(Front fork pin kit, pair)	1
6	1532034	ENDE PROPP COMPACT CHASSI M SKRUER	1
7	1532035-0004	(Fixation parts for front fork castor high or low mounted, black anodized, Compact, pair)	1
8	1532037	BRAKKET TIL GAFFELFESTE COMPACT KUSCHALL	1
9	1563141	(Front fork castor adjusting parts, pair)	1
10	SP1603787-0004	(Front fork castor left, black anodized)	1
11	SP1603788-0004	(Front fork castor right, black anodized)	1

(Front wheels)

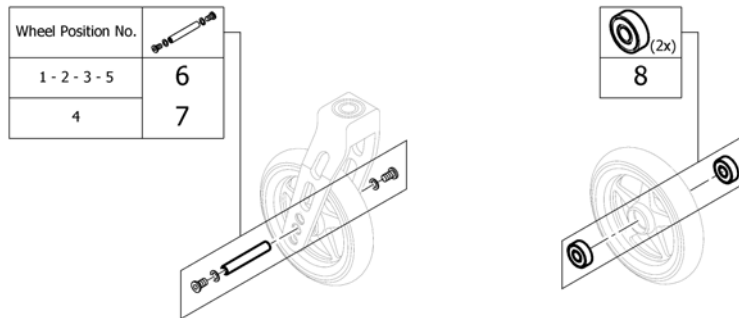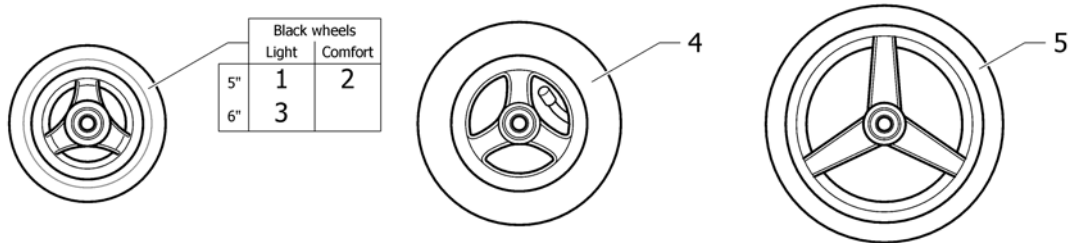


1	156267	SVINGHJUL 5 EVERYDAY 3 EIKER SORT	2
2	1542640	SVINGHJUL 5 SORT 4 CM KUSCHALL COMFORT	2
3	1562628	SVINGHJUL 6 EVERYDAY 3 EIKER SORT GUMMI	2
4	1513189	SVINGHJUL 6"LUFTFYLT KUSCHALL	2
5	1513192	SVINGHJUL 7 SPORT KUSCHALL	2
6	1570108	AKSELSETT SVINGHJUL U DISTANSESKIVER	2
7	1570109	AKSELSETT 6" LUFT KUSCHALL	2
8	1570105	KULELAGER PAR KUSCHALL 6000-2RS	2

(REAR WHEELS)

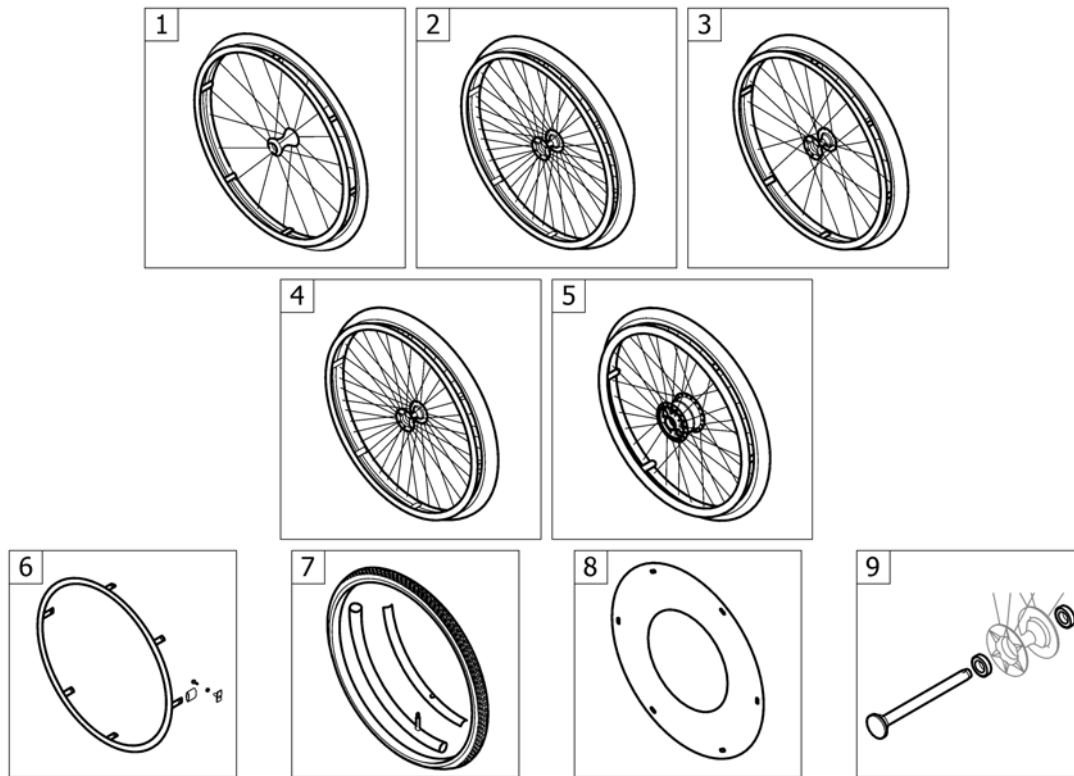


1	-	(Active wheel)	1
2	-	(24" Küschall rear wheels complete)	1
3	-	(High Performance wheel)	1
4	-	(Spinergy LXX wheel)	1
5	-	(Drumbrake wheel)	1
6	-	(Handrim)	1
7	-	(Tyre)	1
8	-	(Spoke guard)	1
9	-	(Quick release axle and bearing)	1

(Active wheel)

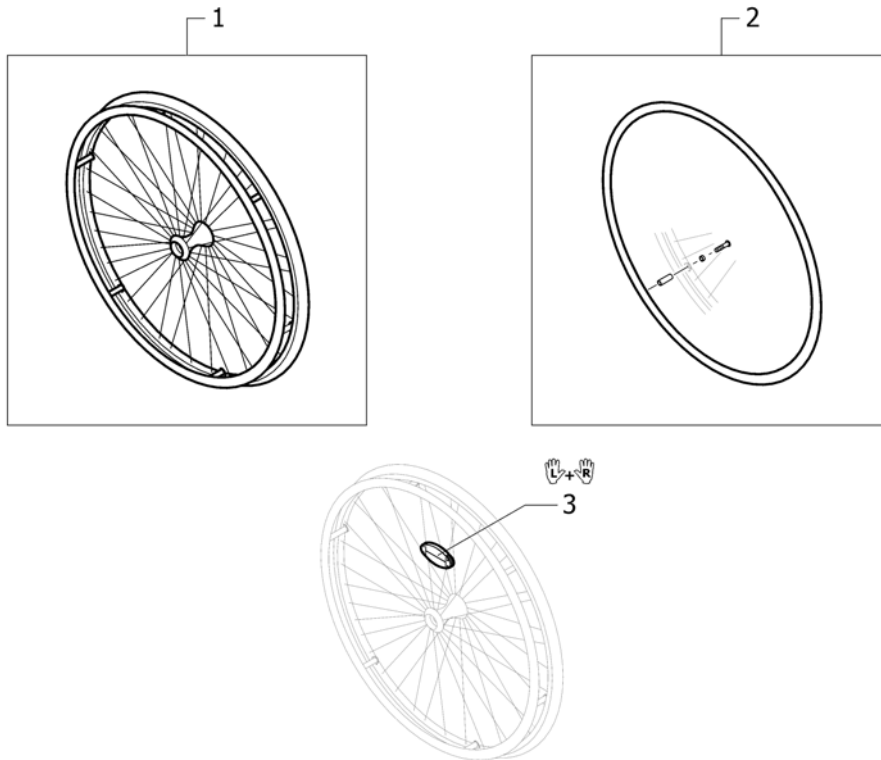


1	1583058	(Rear wheel 24" active)	2
2	1583059	(Handrim 24" Alu incl. Fixation kit for rear wheel 24" Economy)	2
3	1571073	(Spokereflection, pair)	1

(24" Küschall rear wheels complete)

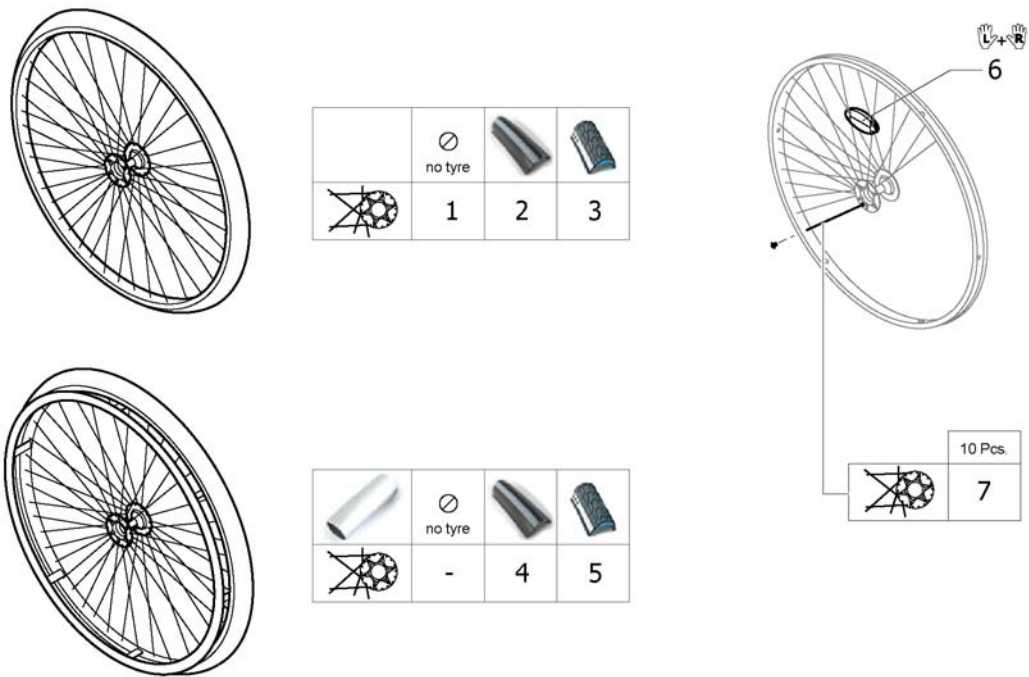


1	1567104-0407	DRIVHJUL 24 STAREC U/DRIVR U/DEKK U/FARGE	2
2	SP1601745	(Rear wheel 24", Starec hub X, without handrim, Rightrun black/grey tyre)	2
3	1554399-0407	(Rear wheel 24", Starec X, colourless, without handrim, marathon evolution)	2
4	SP1601747	(Rear wheel 24", Starec hub X, handrim, Rightrun black/grey tyre)	2
5	1554400-0407	(Rear wheel 24", Starec X, colourless, handrim, marathon evolution)	2
6	1571073	(Spokereflection, pair)	1
7	1579086	(Spokes 253mm complete, kit 10 Pcs.)	1

(High Performance wheel)

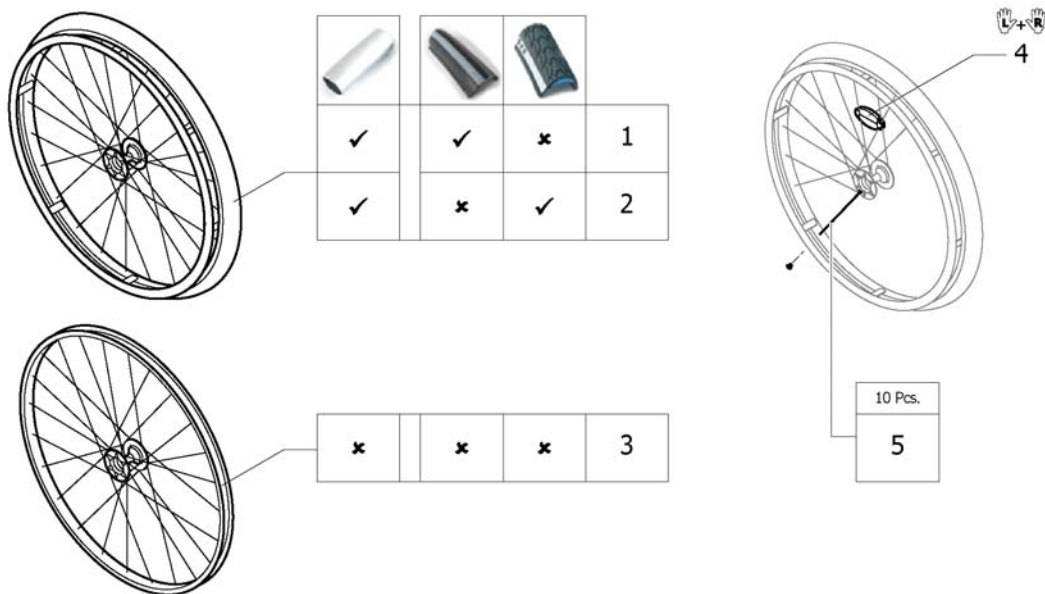


1	SP1601754-0004	(Rear wheel 24" High Performance black anodized, handrim, Rightrun black/ grey)	2
2	1527046-0004	DRIVHJUL 24 HØY PERMANCE SORT DRIV ALU DEKK ME	2
3	1517723-0004	DRIVHJUL 24 HØY PERMANCE SVART UTEN DEKK OG DRIVR	2
4	1571073	(Spokereflection, pair)	1
5	1578867	(Spokes kit for High Performance wheel 24", 10 Pcs.)	1

(Spinergy LXX wheel)

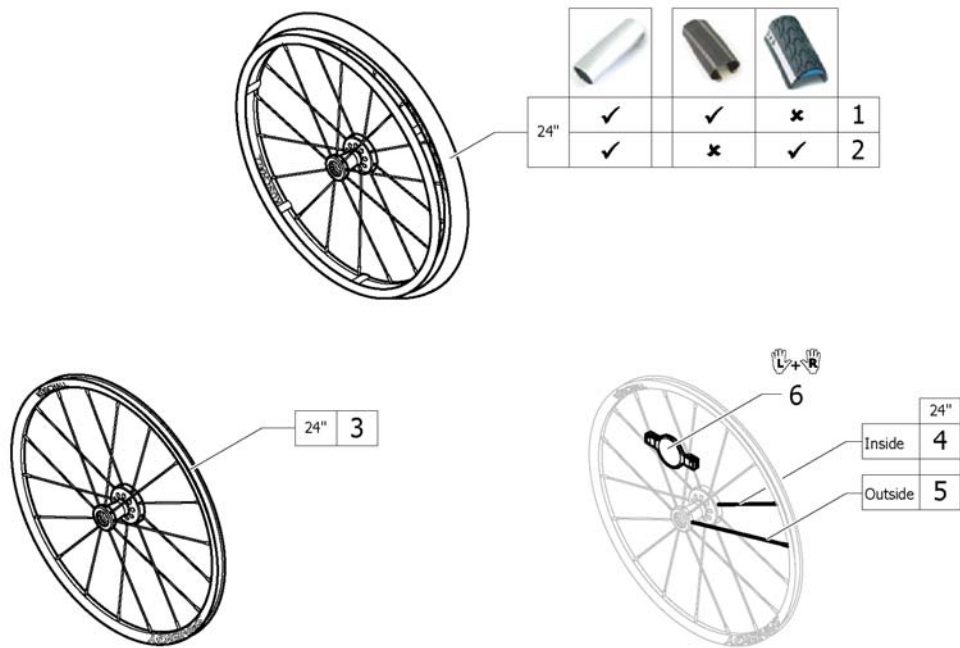

1	1585007	(Rear wheel 24", Spinergy LXX, handrim alu, tyre Schwalbe One)	2
2	1585008	(Rear wheel 24", Spinergy LXX, handrim alu, tyre Marathon Evolution)	2
3	1579102	DRIVHJUL 24 Spinergy LXX UTEN DEKK OG DRIVRING	2
4	1579103	(Spoke inside for Spinergy LXX wheel 24", 236mm., 1 Pcs.)	9
5	1579104	(Spoke outside for Spinergy LXX wheel 24", 249mm., 1 Pcs.)	9
6	1571088	(Spoke reflection for Spinergy wheel, pair)	1


(Drumbrake wheel)

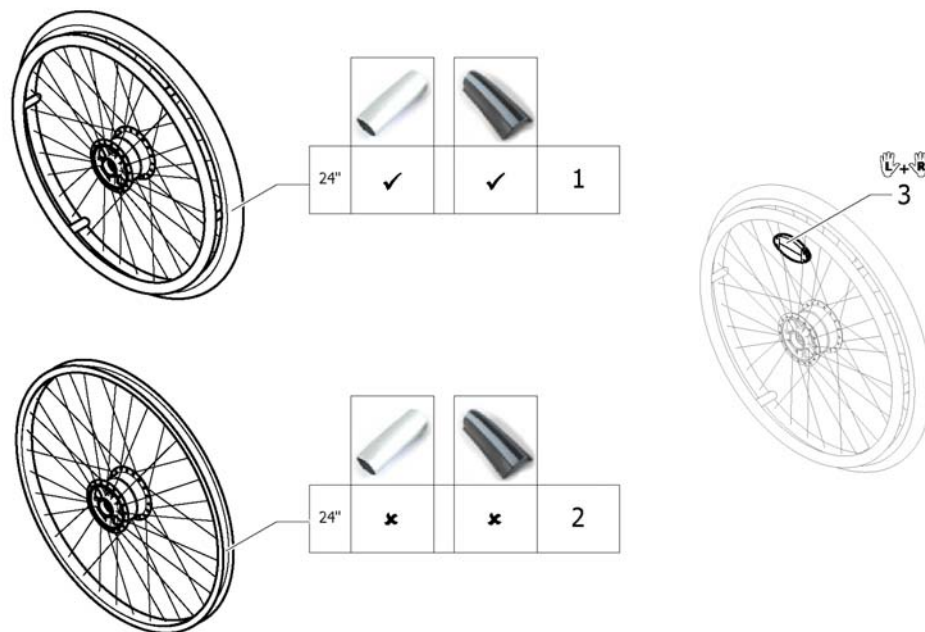

1	SP1604974	(Rear wheel 24" Drumbrake, handrim, Rightrun black/grey)	2
2	SP1604975	(Rear wheel 24" Drumbrake, without handrim, without tyre)	2
3	1571073	(Spokereflection, pair)	1




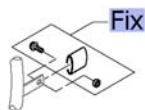


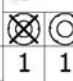
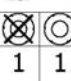




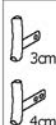


(Handrim)


		24"	24"	24"	24"	24"
 Küschall wheel	3cm	1	2	3	4	5
	4cm	-	-	-	4	-
 High Performance	3cm	1	2	3	4	5
	4cm	-	-	-	4	-
 Spinergy LXX	3cm	1	2	3	4	5
	4cm	-	-	-	4	-
 Drumbrake	3cm	1	2	3	4	5
	4cm	-	-	-	4	-



Fix	-	(Fixation handrim)	2
1	1568714-0005	DRIVRING 24 ALU ELOKSERT KÜSCHALL st.SPINERGY	2
2	1568716	(Handrim 24" Titanium, distance 3cm)	2
3	1568715	DRIVRING 24 STÅL 3CM DIST	2
4	1568703	DRIVRING 24 SUPERGRIP KUSCHALL	2
5	1556916	DRIVRING 24 SURGE KUSCHALL	2





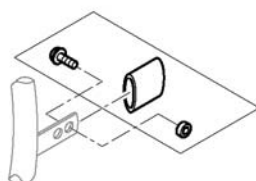




(Fixation handrim)

		Aluminium	Titanium	Steel	Supergrip	Surge			
	3cm								
	4cm								
	3cm	1	1	1	1	1	3	3	
	4cm	-	-	-	-	2	3	-	-
	3cm	4	4	4	4	4	4	6	4
	4cm	-	-	-	-	5	6	-	-
	3cm	4	-	4	-	4	-	6	-
	4cm	-	-	-	-	5	-	-	-
	3cm	1	1	1	1	1	1	3	3
	4cm	-	-	-	-	-	2	3	-



1	1495958	FESTESETT DRIVRING 3CM KUSHALL	2
2	1495959	DRIVRINGSFESTE KIT STD KUSCGHALL HJUL 4CM	2
3	1495961	(Fixation kit handrim, 4 cm with spokes guard)	2
4	1496019	FIKSERINGSKIT FOR DRIVRING HIGH PERFORM 3	2
5	1496020	(Fixation kit handrim for High Performance wheel, 4 cm)	2
6	1497111	(Fixation kit handrim for High Performance wheel, 4 cm with spokes guard)	2










(Tyre)



				
	24"	24"	24"	24"
Active wheels 	1	2	3	4
Kuschall wheels 	1	2	3	4
High Performance 	1	2	3	-
Spinergy LxK 	1	2	3	-
Drumbrake 	1	2	3	4




Acc.	-	(Inner tube and rim protector)	2
1	SP1590118	DEKK 24 Rightrun SORT/GRÅ KUSCHALL	2
2	1570838	DEKK 24 MARATHON KUSHALL PKT SIKKER	2
3	1578802	DEKK 24 SCWALBE SPEEDRUN 24" KUSCHALL	2
4	1570831	DEKK 24 MASSIV GRÅ GROVT MØNSTER	2

Küschall® Compact attract | (REAR WHEELS)
(Inner tube and rim protector)







					
		24"	24"	24"	24"
 Active wheels	A	1	1	1	-
	B	2	2	2	2
 Küschall wheels	A	1	1	1	-
	B	2	2	2	2
 High Performance	A	1	1	1	-
	B	3	3	3	-
 Spinerger LxK	A	1	1	1	-
	B	3	3	3	-
 Drumbrake	A	1	1	1	-
	B	2	2	2	2

 Inner Tube (A)  Rim protector (B)




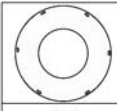



 4

1	1570849	SLANGE 24"X1	2
2	1570845	(Rim protection 24" Large)	2
3	1495962	FELG BAND HIGH PERFORMANCE DEKK KUSCHAL	2
4	1571072	(Rear wheel inset anti-flat for 20,22,24,26" tyre)	1

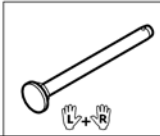(Spoke guard)



	
	24"
Active wheels 	-
Küschall wheels 	1
High Performance 	1
Spinergy LXX 	-
Drumbrake 	1

Küschall® Compact attract | (REAR WHEELS)
(Quick release axle and bearing)

		
	L+R	L+R
 Active wheels	1	-
 Küschall wheels	2	5
 High Performance	1	5
 Spinergy LXX	3	5
 Drumbrake	4	-


1	1509137-0004	HURTIGKOBBLINGSBOLT L 128 M SORT KNAPP KUSCHAL PAR		1
2	1509137-0407	HURTIGKOBBLINGSBOLT L 128 FARGELØS KNAPP KUSCH.PAR		1
3	1507627	QR AKSEL FOR SPINERGY HJUL PAR		1
4	SP1606165-0407	(Quick release axles for drumbrake with cap, pair, colourless)	08.03.2016 →	1
5	1571069	(Bearing R8-2RS 1/2"/1.1/8"x5/16", pair)		2

(BRAKES)

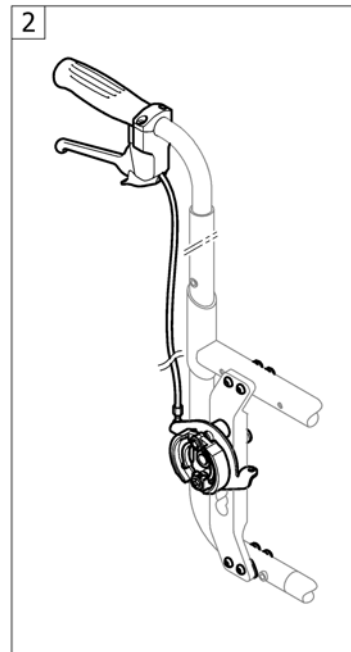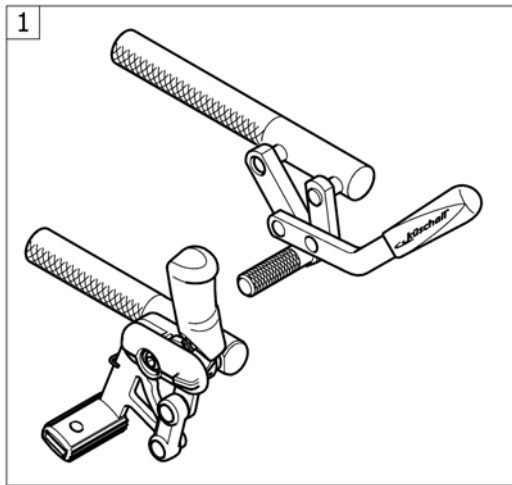


1	-	(Brakes)	1
2	-	(Drum brake)	1

(Brakes)






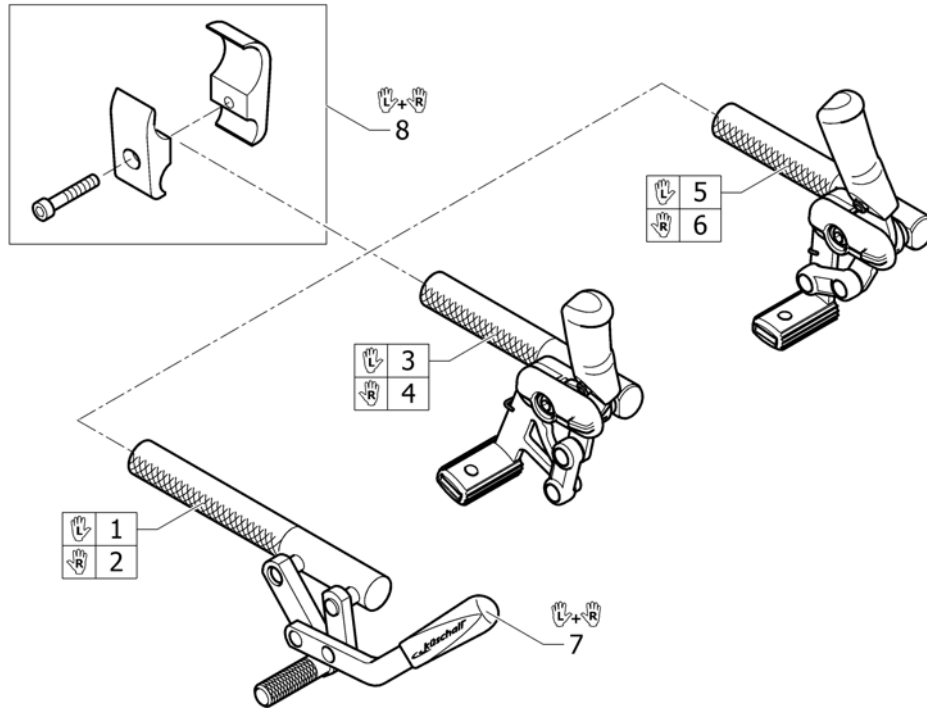


1	1512129	BREMS DRA FOR Å LÅSE VE STD ULCPAKID FRA 150208	1
2	1512130	BREMS DRA FOR Å LÅSE HØ STD ULCPAKID FRA 150208	1
3	SP1604007	(Attract Pull brake left)	1
4	SP1604008	(Attract Pull brake right)	1
5	SP1604009	(Attract Push brake left)	1
6	SP1604010	(Attract Push brake right)	1
7	1513177	HÅNDTAK BREMS PAR KUSCHALL	1
8	1513176	KLEMME BREMS KPL K-SERIE CHAMP 110708	2

(Drum brake)

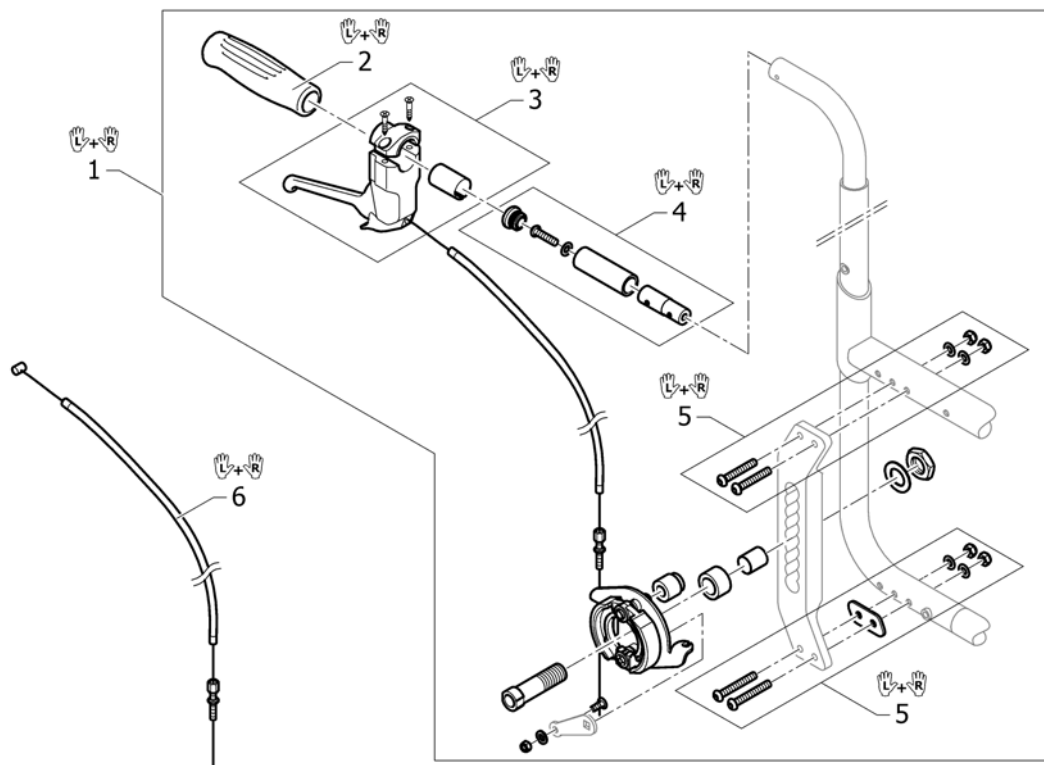


1	SP1604976	(Drumbrake set, pair)	1
2	1507734	KJØREHÅNDTAK SVART PLAST PAR KUSCHALL	1
3	SP1604978	(Drumbrake handle, pair)	1
4	1527108	KØREHANDTAKS FORLENGER ASSMAN BREMS KUSCHALL	1
5	SP1604979	(Fixation kit for Vertical brace with drumbrake, pair)	1
6	SP1604977	(Drumbrake cable with hose, pair)	1

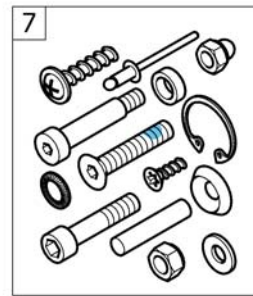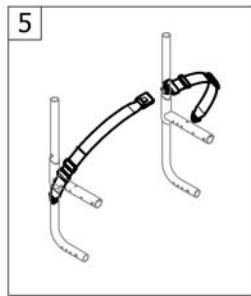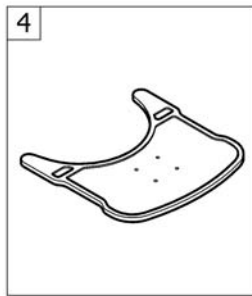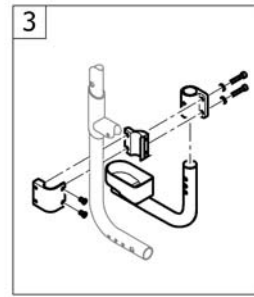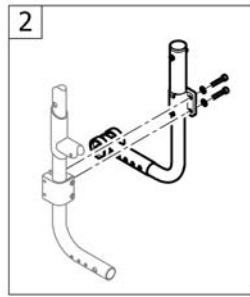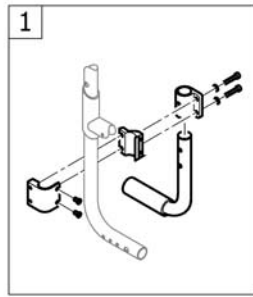


1	-	(Tipper aid)	1
2	-	(Antitipper)	1
3	-	(Cane holder)	1
4	-	(Therapy table)	1
5	-	(Seat belt)	1
6	-	(Chair accessories)	1
7	-	(Fixation: screws, ...)	1

(Tipper aid)

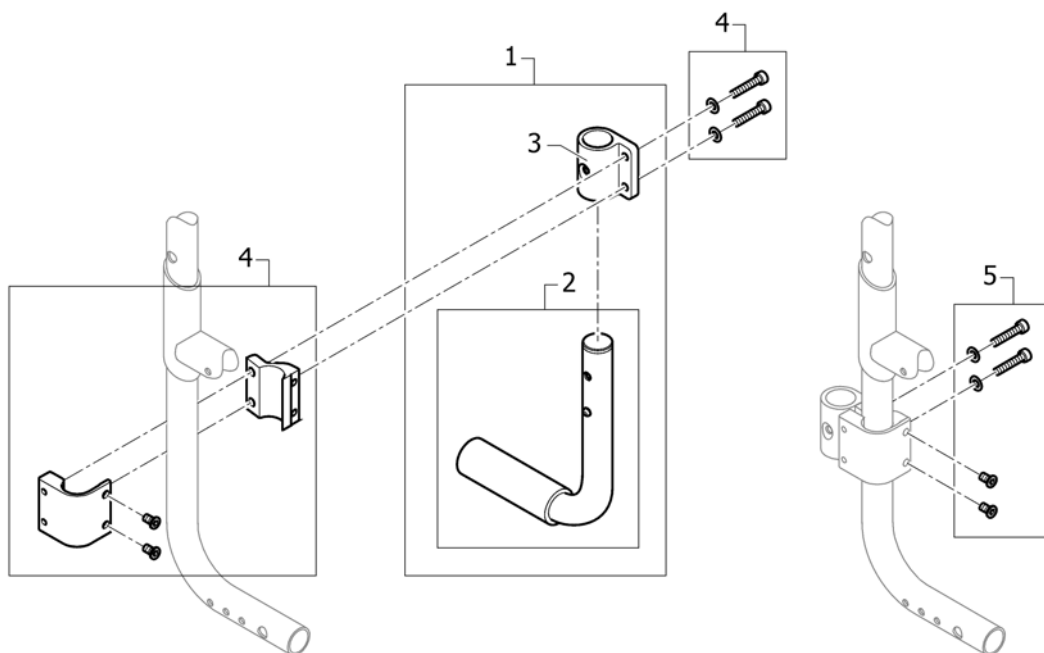


1	SP1604987	(Tipper-Aid with holder)	1
2	SP1604988	(Tipper-Aid)	1
3	SP1604984	(Holder for adaption)	1
4	SP1604985	(Clamping piece to adaption)	1
5	SP1604986	(Fixation kit for clamping piece to adaption)	1

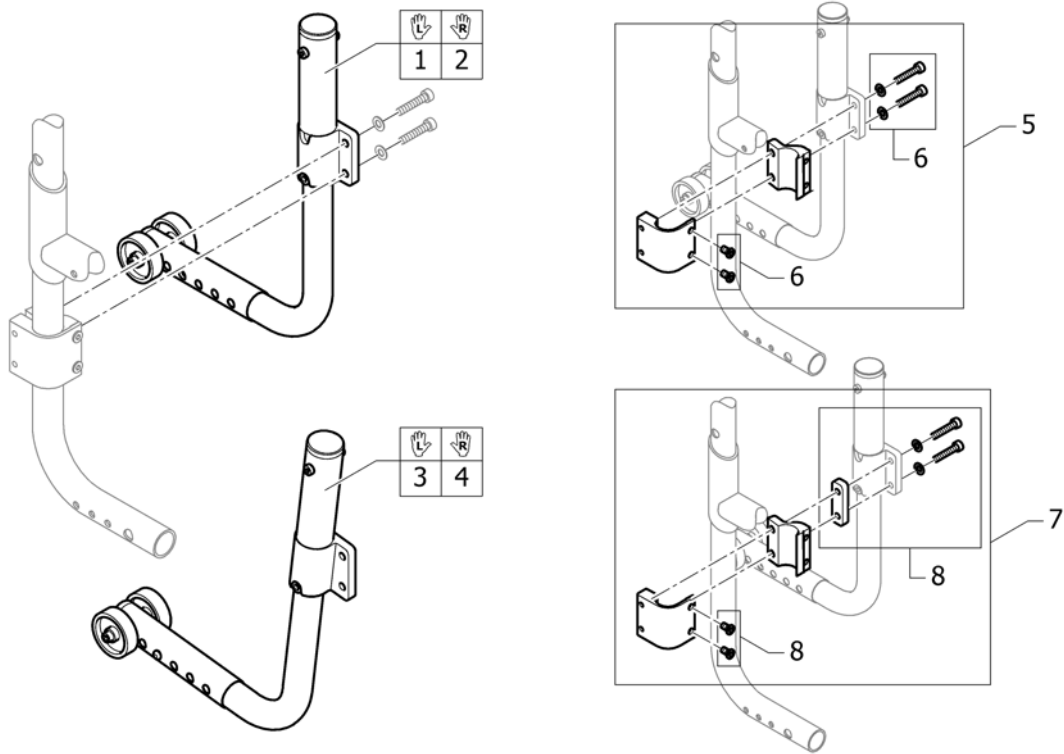


1	SP1604989	(Antitipper, left)	1
2	SP1604990	(Antitipper, right)	1
3	SP1604991	(Antitipper 7°, left)	1
4	SP1604992	(Antitipper 7°, right)	1
5	SP1604985	(Clamping piece to adaption)	1
6	SP1604986	(Fixation kit for clamping piece to adaption)	1
7	SP1604993	(Clamping piece to adaption with spacer) <i>Needed in combination with siderest with desk function</i>	1
8	SP1604994	(Fixation kit for clamping piece to adaption with spacer) <i>Needed in combination with siderest with desk function</i>	1

(Cane holder)

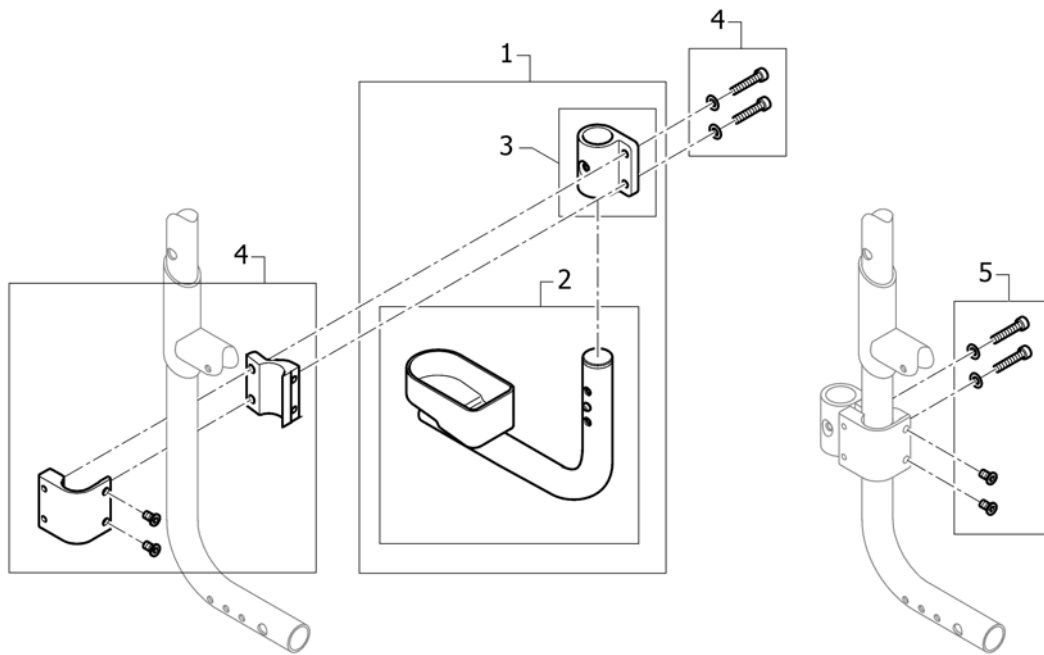


1	SP1604982	(cane holder with holder)	1
2	SP1604983	(cane holder)	1
3	SP1604984	(Holder for adaption)	1
4	SP1604985	(Clamping piece to adaption)	1
5	SP1604986	(Fixation kit for clamping piece to adaption)	1

(Therapy table)

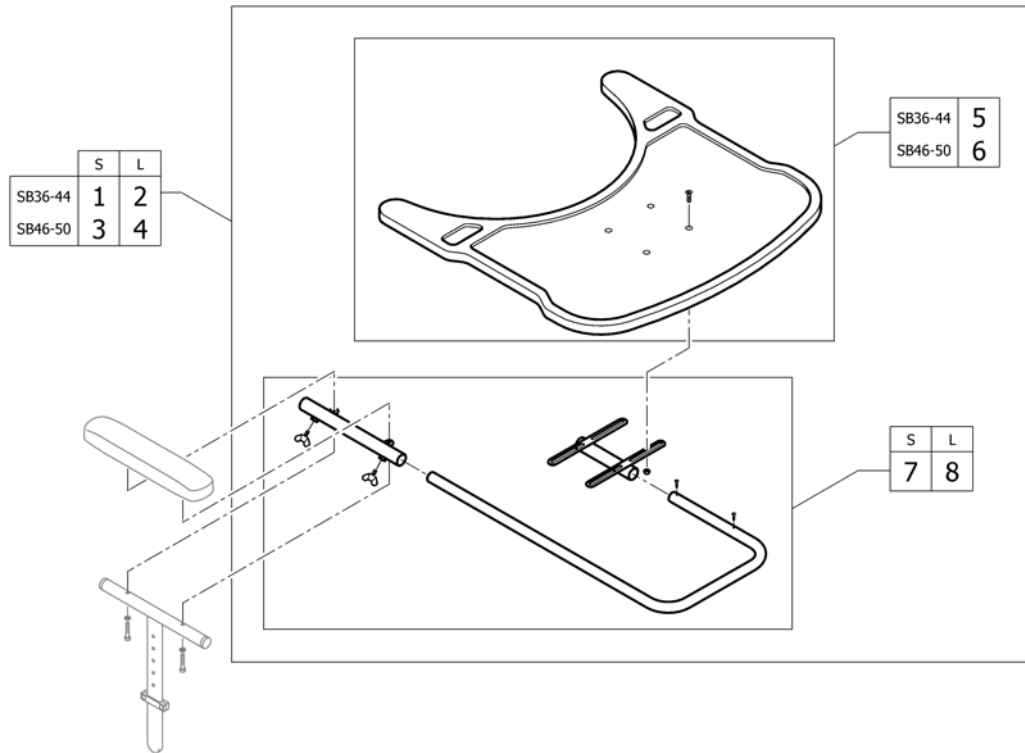


1	1439492	BORD MEDIUM KUSCHALL CO SB36-44 KORT POLSTER	1
2	1438374	BORD TERAPI M KPL SB 36-44 LANG POLSTER	1
3	1439493	BORD TERAPI L KPL FOR ARMLEN KORT	1
4	1438375	BORD TERAPI L KPL TIL SB 46-50 LANG POLSTER	1
5	1582670	(Therapy table M)	1
6	1582671	(Therapy table L)	1
7	1582672	(Fixation kit for therapy table on armrest short)	1
8	1582673	(Fixation kit for therapy table on armrest long)	1

(Seat belt)

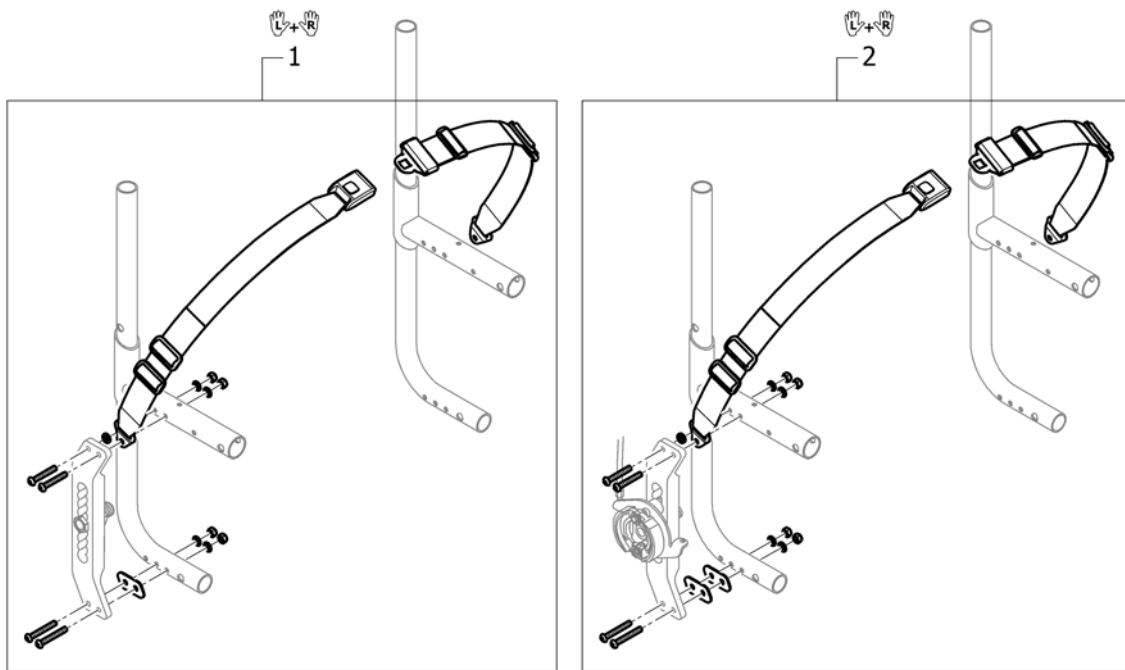


1	SP1604980	(Seat belt)	1
2	SP1604981	(Seat belt mounted with drumbrake wheel)	1

Küschall® Compact attract | (OPTIONS & ACCESSORIES)
(Chair accessories)

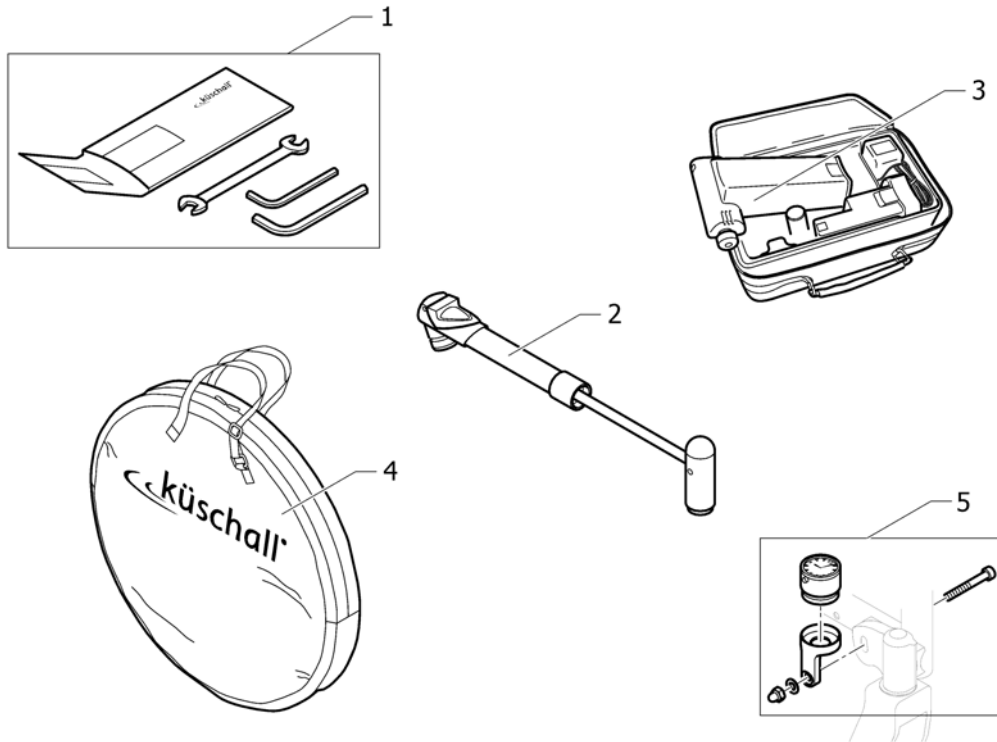


1	1518285	VERKTØYSETT KUSCHALL	1
2	1517924	PUMPE KUSCHALL MANUELL	1
3	1518286	(Air man)	1
4	1518287	VESKE FOR 24 OG 26 HJUL KÜSCHALL LOGO	1
5	SP1601738	(Küschall watch mounted on front fork castor, kit)	1

(Fixation: screws, ...)

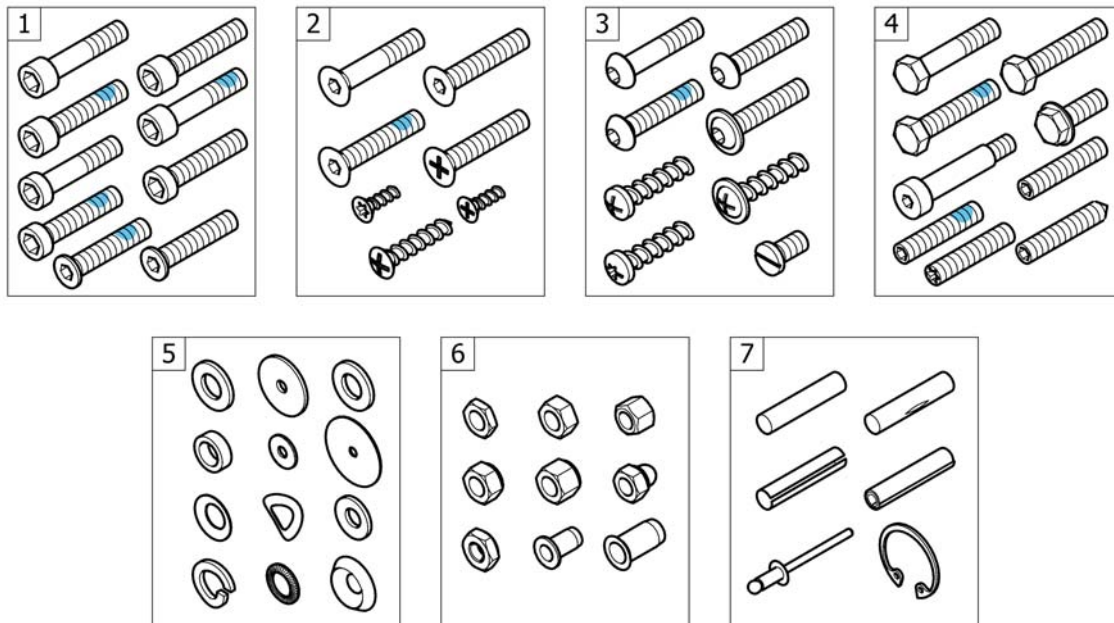

1	-	(Socket head cap screws)	1
2	-	(Countersunk screws)	1
3	-	(Button head screws)	1
4	-	(Hex-head screws, diverse, ...)	1
5	-	(Washers)	1
6	-	(Nuts)	1
7	-	(Pins, rivets, ...)	1

(Socket head cap screws)

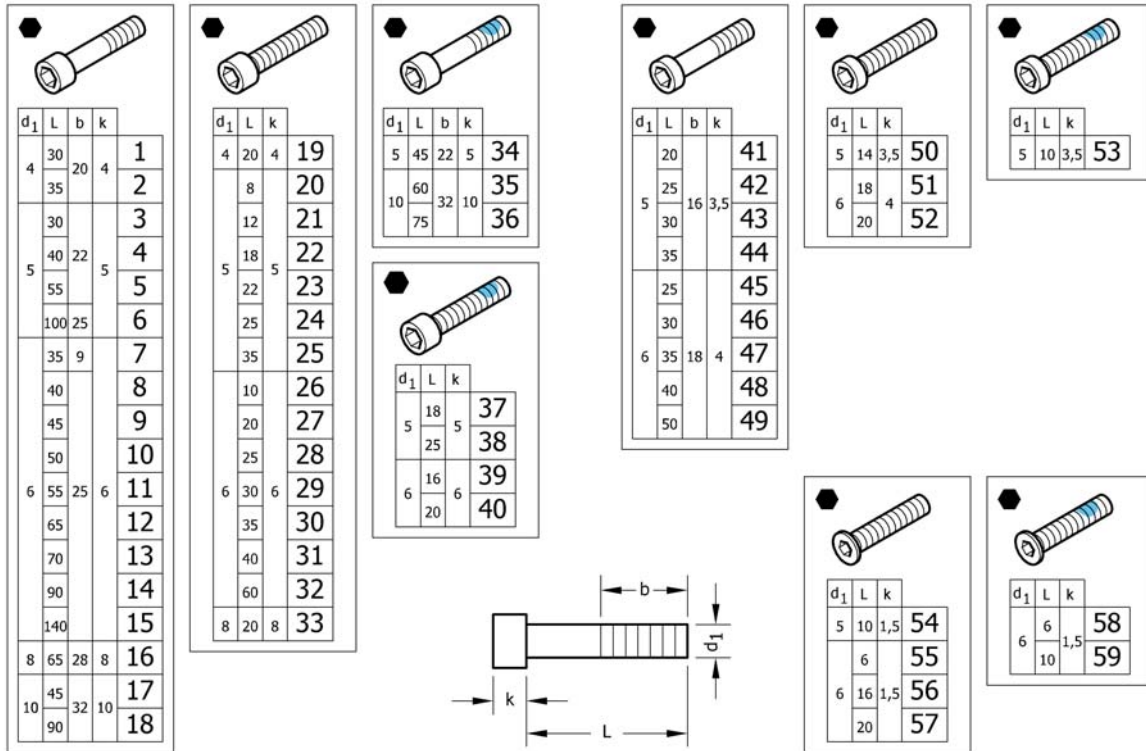

1	1579115	(Hex socket head cap screws partially threaded, zi.Blue, M4x30, kit 25 Pcs.) <i>Replaces 1560191 (25x)</i>	1
2	1579116	(Hex socket head cap screws partially threaded, zi.Blue, M4x35, kit 25 Pcs.) <i>Replaces 1563765 (25x)</i>	1
3	1579117	(Hex socket head cap screws partially threaded, black, M5x30, kit 25 Pcs.) <i>Replaces 100.1091 (25x)</i>	1
4	1579118	(Hex socket head cap screws partially threaded, black, M5x40, kit 25 Pcs.) <i>Replaces 100.1093 (25x)</i>	1
5	1579119	(Hex socket head cap screws partially threaded, black, M5x55, kit 25 Pcs.) <i>Replaces 100.1096 (25x)</i>	1
6	1579120	(Hex socket head cap screws partially threaded, black, M5x100, kit 10 Pcs.) <i>Replaces 1425240 (10x)</i>	1
7	1579121	(Hex socket head cap screws partially threaded, black, M6x35, kit 5 Pcs.) <i>Replaces 107.1001 (5x)</i>	1
8	1579122	(Hex socket head cap screws partially threaded, black, M6x40, kit 25 Pcs.) <i>Replaces 100.1133 (25x)</i>	1
9	1579123	(Hex socket head cap screws partially threaded, black, M6x45, kit 25 Pcs.) <i>Replaces 100.1134 (25x)</i>	1
10	1579124	(Hex socket head cap screws partially threaded, black, M6x50, kit 25 Pcs.) <i>Replaces 100.1135 (25x)</i>	1
11	1579125	(Hex socket head cap screws partially threaded, black, M6x55, kit 25 Pcs.) <i>Replaces 100.1136 (25x)</i>	1
11	1579126	(Hex socket head cap screws partially threaded, zi.Blue, M6x55, kit 25 Pcs.) <i>Replaces ZM006551A (25x)</i>	1
12	1579127	(Hex socket head cap screws partially threaded, black, M6x65, kit 10 Pcs.) <i>Replaces 100.1138 (10x)</i>	1
13	1579128	(Hex socket head cap screws partially threaded, black, M6x70, kit 10 Pcs.) <i>Replaces 100.1139 (10x)</i>	1
14	1579129	(Hex socket head cap screws partially threaded, black, M6x90, kit 10 Pcs.) <i>Replaces 1529118 (10x)</i>	1
15	1579130	(Hex socket head cap screws partially threaded, black, M6x140, kit 10 Pcs.) <i>Replaces 1529117 (10x)</i>	1


(Socket head cap screws)

16	1579131	(Hex socket head cap screws partially threaded, black, M8x65, kit 10 Pcs.) <i>Replaces 1510316 (10x)</i>	1
17	1579132	(Hex socket head cap screws partially threaded, black, M10x45, kit 2 Pcs.) <i>Replaces 1496010 (2x)</i>	1
18	1579133	(Hex socket head cap screws partially threaded, black, M10x90, kit 2 Pcs.) <i>Replaces 1496013 (2x)</i>	1
19	1579134	(Hex socket head cap screws fully threaded, black, M4x20, kit 25 Pcs.) <i>Replaces 100.1050 (25x)</i>	1
20	1579135	(Hex socket head cap screws fully threaded, black, M5x8, kit 5 Pcs.) <i>Replaces 1493753 (5x)</i>	1
21	1579136	(Hex socket head cap screws fully threaded, black, M5x12, kit 25 Pcs.) <i>Replaces 100.1084 (25x)</i>	1
22	1579137	(Hex socket head cap screws fully threaded, black, M5x18, kit 25 Pcs.) <i>Replaces 100.1087 (25x)</i>	1
23	1579138	(Hex socket head cap screws fully threaded, black, M5x22, kit 25 Pcs.) <i>Replaces 100.1089 (25x)</i>	1
24	1579139	(Hex socket head cap screws fully threaded, black, M5x25, kit 25 Pcs.) <i>Replaces 100.1090 (25x)</i>	1
25	1579140	(Hex socket head cap screws fully threaded, black, M5x35, kit 25 Pcs.) <i>Replaces 100.1092 (25x)</i>	1
26	1579141	(Hex socket head cap screws fully threaded, black, M6x10, kit 25 Pcs.) <i>Replaces 100.1123 (25x)</i>	1
27	1579142	(Hex socket head cap screws fully threaded, black, M6x20, kit 25 Pcs.) <i>Replaces 100.1128 (25x)</i>	1
28	1579143	(Hex socket head cap screws fully threaded, black, M6x25, kit 25 Pcs.) <i>Replaces 100.1130 (25x)</i>	1
29	1579144	(Hex socket head cap screws fully threaded, black, M6x30, kit 25 Pcs.) <i>Replaces 100.1131 (25x)</i>	1
30	1579145	(Hex socket head cap screws fully threaded, black, M6x35, kit 25 Pcs.) <i>Replaces 100.1132 (25x)</i>	1
31	1579146	(Hex socket head cap screws fully threaded, black, M6x40, kit 10 Pcs.) <i>Replaces 1563349 (10x)</i>	1
32	1579147	(Hex socket head cap screws fully threaded, black, M6x60, kit 10 Pcs.) <i>Replaces 100.1137 (10x)</i>	1
33	1579148	(Hex socket head cap screws fully threaded, black, M8x20, kit 10 Pcs.) <i>Replaces 1510307 (10x)</i>	1
34	1579149	(Hex sock.Head cap screws partial.Thread., black, TufLok, M5x45, kit 5 Pcs.) <i>Replaces 1496014 (5x)</i>	1
35	1579150	(Hex sock.Head cap screws partial.Thread., black, TufLok, M10x60, kit 2 Pcs.) <i>Replaces 1496011 (2x)</i>	1
36	1579151	(Hex sock.Head cap screws partial.Thread., black, TufLok, M10x75, kit 2 Pcs.) <i>Replaces 1496012 (2x)</i>	1
37	1579152	(Hex socket head cap screws fully threaded, black, TufLok, M5x18, kit 10 Pcs.) <i>Replaces 1425241 (10x)</i>	1
38	1579153	(Hex socket head cap screws fully threaded, black, TufLok, M5x25, kit 10 Pcs.) <i>Replaces 1570153 (10x)</i>	1
39	1579154	(Hex socket head cap screws full.Thread., zi.Blue, TufLok, M6x16, kit 10 Pcs.) <i>Replaces 1560219 (10x)</i>	1
40	1579155	(Hex socket head cap screws fully threaded, black, TufLok, M6x20, kit 10 Pcs.) <i>Replaces 1498775 (10x)</i>	1
41	1579156	(Hex sock.Head cap screws, low head, part.Thread., black, M5x20, kit 25 Pcs.) <i>Replaces 1438501 (25x)</i>	1
42	1579157	(Hex sock.Head cap screws, low head, part.Thread., black, M5x25, kit 25 Pcs.) <i>Replaces 1438518 (25x)</i>	1
43	1579158	(Hex sock.Head cap screws, low head, part.Thread., black, M5x30, kit 25 Pcs.) <i>Replaces 100.5090 (25x)</i>	1

(Socket head cap screws)

44	1579159	(Hex sock.Head cap screws, low head, part.Thread., black, M5x35, kit 25 Pcs.) <i>Replaces 100.5091 (25x)</i>	1
45	1579160	(Hex sock.Head cap screws, low head, part.Thread., black, M6x25, kit 25 Pcs.) <i>Replaces 1432120 (25x)</i>	1
46	1579161	(Hex sock.Head cap screws, low head, part.Thread., black, M6x30, kit 25 Pcs.) <i>Replaces 100.5130 (25x)</i>	1
47	1579162	(Hex sock.Head cap screws, low head, part.Thread., black, M6x35, kit 25 Pcs.) <i>Replaces 1438504 (25x)</i>	1
48	1579163	(Hex sock.Head cap screws, low head, part.Thread., black, M6x40, kit 25 Pcs.) <i>Replaces 100.5132 (25x)</i>	1
49	1579164	(Hex sock.Head cap screws, low head, part.Thread., black, M6x50, kit 25 Pcs.) <i>Replaces 100.5134 (25x)</i>	1
50	1579165	(Hex sock.Head cap screws, low head, part.Thread., black, M5x14, kit 25 Pcs.) <i>Replaces 100.5085 (25x)</i>	1
51	1579166	(Hex sock.Head cap screws, low head, part.Thread., black, M6x18, kit 25 Pcs.) <i>Replaces 100.5126 (25x)</i>	1
52	1579167	(Hex sock.Head cap screws, low head, part.Thread., black, M6x20, kit 25 Pcs.) <i>Replaces 100.5127 (25x)</i>	1
53	1579168	(Hex soc.Head cap scr., low head, part.Threa., bl., TufL., M5x10, kit 10 Pcs.) <i>Replaces 1496026 (10x)</i>	1
54	1579169	(Hex socket head cap screws, special low head, black, M5x10, kit 25 Pcs.) <i>Replaces 1437290 (25x)</i>	1
55	1579170	(Hex socket head cap screws, special low head, black, M6x6, kit 25 Pcs.) <i>Replaces 1425182 (25x)</i>	1
56	1579171	(Hex socket head cap screws, special low head, black, M6x16, kit 25 Pcs.) <i>Replaces 1541181 (25x)</i>	1
57	1579172	(Hex socket head cap screws, special low head, black, M6x20, kit 25 Pcs.) <i>Replaces 107.1022 (25x)</i>	1
58	1579173	(Hex socket head cap screws, spec.Low head, black, TufLok, M5x6, kit 10 Pcs.) <i>Replaces 1532039 (10x)</i>	1
59	1579174	(Hex socket head cap screws, spec.Low head, black, TufLok, M6x10, kit 10 Pcs.) <i>Replaces 1420946 (10x)</i>	1

(Countersunk screws)

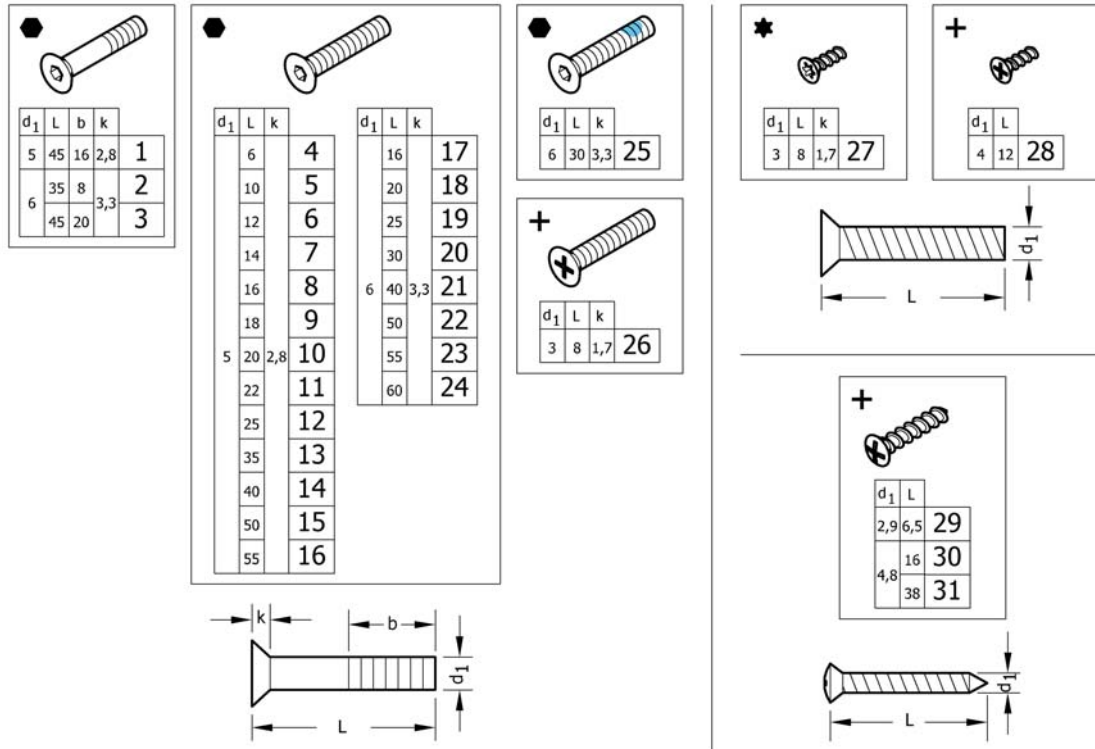


1	1579175	(Countersunk head screws partially threaded, Black, M5x45, kit 10 Pcs.) <i>Replaces 1438773 (10x)</i>	1
2	1579176	(Countersunk head screws partially threaded, Black, M6x35, kit 10 Pcs.) <i>Replaces 107.1004 (10x)</i>	1
3	1579177	(Countersunk head screws partially threaded, Black, M6x45, kit 10 Pcs.) <i>Replaces 100.3133 (10x)</i>	1
4	1579178	(Countersunk head screws fully threaded, Black, M5x6, kit 25 Pcs.) <i>Replaces 1445273 (25x)</i>	1
5	1579179	(Countersunk head screws fully threaded, Black, M5x10, kit 25 Pcs.) <i>Replaces 100.3082 (25x)</i>	1
6	1579180	(Countersunk head screws fully threaded, Black, M5x12, kit 25 Pcs.) <i>Replaces 1438999 (25x)</i>	1
7	1579181	(Countersunk head screws fully threaded, Black, M5x14, kit 25 Pcs.) <i>Replaces 107.1015 (25x)</i>	1
8	1579182	(Countersunk head screws fully threaded, Black, M5x16, kit 25 Pcs.) <i>Replaces 100.3085 (25x)</i>	1
9	1579183	(Countersunk head screws fully threaded, Black, M5x18, kit 25 Pcs.) <i>Replaces 100.3086 (25x)</i>	1
10	1579184	(Countersunk head screws fully threaded, Black, M5x20, kit 25 Pcs.) <i>Replaces 100.3087 (25x)</i>	1
11	1579185	(Countersunk head screws fully threaded, Black, M5x22, kit 25 Pcs.) <i>Replaces 1416930 (25x)</i>	1
12	1579186	(Countersunk head screws fully threaded, Black, M5x25, kit 25 Pcs.) <i>Replaces 100.3089 (25x)</i>	1
13	1579187	(Countersunk head screws fully threaded, Black, M5x35, kit 25 Pcs.) <i>Replaces 100.3091 (25x)</i>	1
14	1579188	(Countersunk head screws fully threaded, Black, M5x40, kit 25 Pcs.) <i>Replaces 1432148 (25x)</i>	1
15	1579189	(Countersunk head screws fully threaded, Black, M5x50, kit 25 Pcs.) <i>Replaces 100.3094 (25x)</i>	1
16	1579190	(Countersunk head screws fully threaded, Black, M5x55, kit 25 Pcs.) <i>Replaces 100.3095 (25x)</i>	1
17	1579191	(Countersunk head screws fully threaded, Black, M6x16, kit 25 Pcs.) <i>Replaces 100.3125 (25x)</i>	1
18	1579192	(Countersunk head screws full.Threaded, zinc flake coat., M6x20, kit 25 Pcs.) <i>Replaces 100.3127 (25x)</i>	1

(Countersunk screws)

18	1579193	(Countersunk head screws fully threaded, Black, M6x20, kit 25 Pcs.) <i>Replaces 1437809 (25x)</i>	1
19	1579194	(Countersunk head screws fully threaded, Black, M6x25, kit 25 Pcs.) <i>Replaces 100.3129 (25x)</i>	1
20	1579195	(Countersunk head screws fully threaded, Black, M6x30, kit 25 Pcs.) <i>Replaces 100.3130 (25x)</i>	1
21	1579196	(Countersunk head screws fully threaded, Black, M6x40, kit 25 Pcs.) <i>Replaces 100.3132 (25x)</i>	1
22	1579197	(Countersunk head screws fully threaded, Black, M6x50, kit 25 Pcs.) <i>Replaces 100.3134 (25x)</i>	1
23	1579198	(Countersunk head screws fully threaded, Black, M6x55, kit 25 Pcs.) <i>Replaces 100.3135 (25x)</i>	1
24	1579199	(Countersunk head screws fully threaded, Black, M6x60, kit 25 Pcs.) <i>Replaces 100.3136 (25x)</i>	1
25	1579200	(Countersunk head screws fully threaded, Black, TufLok, M6x30, kit 25 Pcs.) <i>Replaces 1550479 (25x)</i>	1
26	1579201	(Countersunk head screws fully threaded, zi.Blue, M3x8, kit 25 Pcs.) <i>Replaces 1529279 (25x)</i>	1
27	1579202	(Countersunk head PT screws, Torx, stainless steel, 3x8, kit 25 Pcs.) <i>Replaces 1529278 (25x)</i>	1
28	1579203	(Countersunk head PT screws, Black, KA40x12, kit 25 Pcs.) <i>Replaces 1438513 (25x)</i>	1
29	1579204	(Countersunk head tapping screws with cone end, Black, ST2,9x6,5, kit 25 Pcs.) <i>Replaces 126.5041 (25x)</i>	1
30	1579205	(Countersunk head tapping screws with cone end, Black, ST4,8x16, kit 25 Pcs.) <i>Replaces 126.5202 (25x)</i>	1
31	1579206	(Countersunk head tapping screws with cone end, Black, ST4,8x38, kit 25 Pcs.) <i>Replaces 126.5207 (25x)</i>	1

(Button head screws)

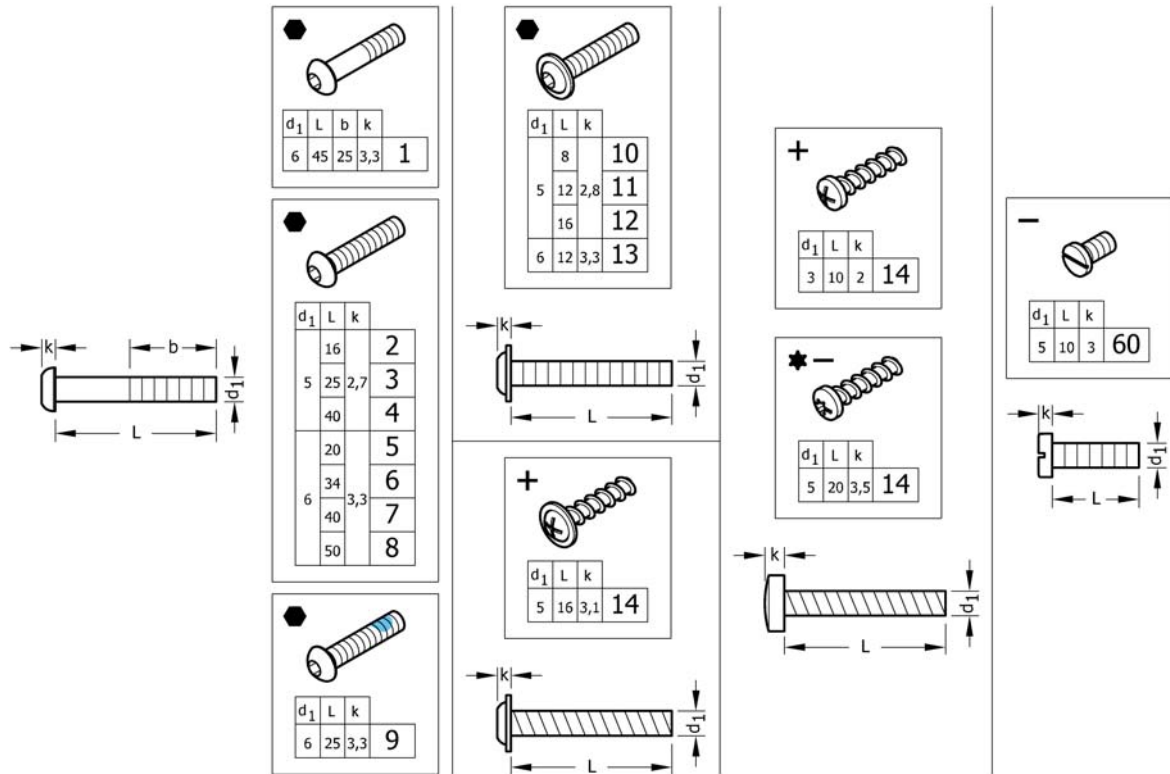


1	1579259	(Hex sock.Button head cap screws partially thread., Black, M6x45, kit 25 Pcs.) <i>Replaces 1438507 (25x)</i>	1
2	1579260	(Hex socket button head cap screws fully threaded, Black, M5x16, kit 25 Pcs.) <i>Replaces 1445341-0003 (25x)</i>	1
3	1579261	(Hex socket button head cap screws fully threaded, Black, M5x25, kit 25 Pcs.) <i>Replaces 107.1021 (25x)</i>	1
4	1579262	(Hex socket button head cap screws fully threaded, Black, M5x40, kit 25 Pcs.) <i>Replaces 1445101-0003 (25x)</i>	1
5	1579263	(Hex socket button head cap screws fully threaded, Black, M6x20, kit 25 Pcs.) <i>Replaces 1439153 (25x)</i>	1
6	1579264	(Hex socket button head cap screws fully threaded, Black, M6x34, kit 25 Pcs.) <i>Replaces 1438506-0003 (25x)</i>	1
7	1579265	(Hex socket button head cap screws fully threaded, Black, M6x40, kit 25 Pcs.) <i>Replaces 1439530-0003 (25x)</i>	1
8	1579266	(Hex socket button head cap screws fully threaded, Black, M6x50, kit 25 Pcs.) <i>Replaces 1439531-0003 (25x)</i>	1
9	1579267	(Hex sock.But.Head cap screws full.Thread., Black, TufLok, M6x25, kit 25 Pcs.) <i>Replaces 1438505 (25x)</i>	1
10	1579268	(Button head socket cap screws with flange, Black, M5x8, kit 25 Pcs.) <i>Replaces 1536268 (25x)</i>	1
11	1579269	(Button head socket cap screws with flange, Black, M5x12, kit 25 Pcs.) <i>Replaces 1529290 (25x)</i>	1
12	1579270	(Button head socket cap screws with flange, Black, M5x16, kit 25 Pcs.) <i>Replaces 1445102-0003 (25x)</i>	1
13	1579271	(Button head socket cap screws with flange, Black, M6x12, kit 25 Pcs.) <i>Replaces 1432204 (25x)</i>	1
14	1579272	(Round washer head PT® screws, form H, zinc plated blue, M5x16, kit 25 Pcs.) <i>Replaces 1536057 (25x)</i>	1

(Button head screws)

15	1579273	(Pan head PT screws, form H, zinc plated blue, M3x10, kit 25 Pcs.) <i>Replaces ZM003510A (25x)</i>	1
16	1579274	(Pan head PT screws, Torx, Black, M5x20, kit 25 Pcs.) <i>Replaces 1424569 (25x)</i>	1
17	1579275	(Slotted machine screws, Black, M5x10, kit 25 Pcs.) <i>Replaces 102.2083 (25x)</i>	1

(Hex-head screws, diverse, ...)

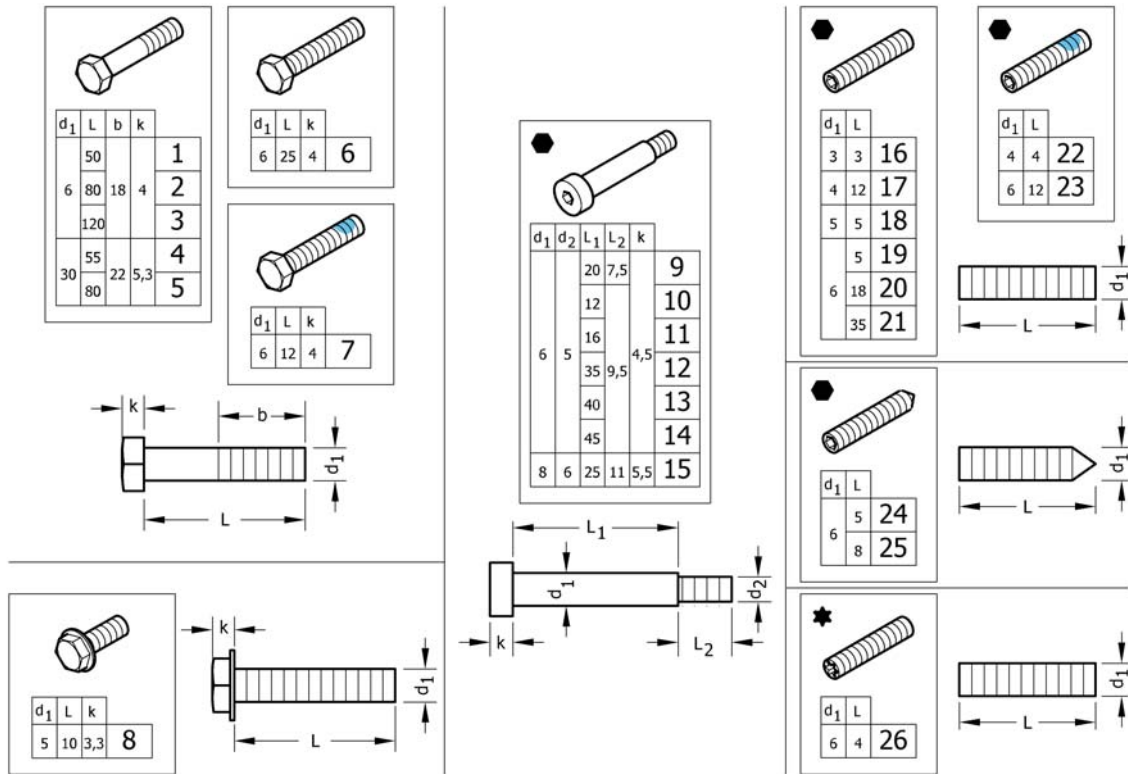


1	1579276	(Hex head bolts partially threaded, Black, M6x50, kit 10 Pcs.) <i>Replaces 101.1134 (10x)</i>	1
2	1579277	(Hex head bolts partially threaded, Black, M6x80, kit 10 Pcs.) <i>Replaces 1438508 (10x)</i>	1
3	1579278	(Hex head bolts partially threaded, Black, M6x120, kit 10 Pcs.) <i>Replaces 1438509 (10x)</i>	1
4	1579279	(Hex head bolts partially threaded, Black, M8x55, kit 10 Pcs.) <i>Replaces 101.1174 (10x)</i>	1
5	1579280	(Hex head bolts partially threaded, Black, M8x80, kit 10 Pcs.) <i>Replaces 101.1179 (10x)</i>	1
6	1579281	(Hex head screws fully threaded, Black, M6x25, kit 10 Pcs.) <i>Replaces 1437299 (10x)</i>	1
7	1579282	(Hex head screws fully threaded, Black, TufLok, M6x12, kit 10 Pcs.) <i>Replaces 1564245 (10x)</i>	1
8	1579283	(Hex flange head machine screws with flangeBlack, M5x10, kit 10 Pcs.) <i>Replaces 107.1014 (10x)</i>	1
9	1579295	(Hex socket head shoulder bolts, Black, Ø6x20 / M5, kit 4 Pcs.) <i>Replaces 1488934-0003 (4x)</i>	1
10	1579296	(Hex socket head shoulder bolts, Black, Ø6x12 / M5, kit 4 Pcs.) <i>Replaces 1564246 (4x)</i>	1
11	1579297	(Hex socket head shoulder bolts, Black, Ø6x16 / M5, kit 4 Pcs.) <i>Replaces 1529291 (4x)</i>	1
12	1579298	(Hex socket head shoulder bolts, Black, Ø6x35 / M5, kit 4 Pcs.) <i>Replaces 1496034 (4x)</i>	1
13	1579299	(Hex socket head shoulder bolts, Black, Ø6x40 / M5, kit 4 Pcs.) <i>Replaces 107.1023 (4x)</i>	1
14	1579300	(Hex socket head shoulder bolts, Black, Ø6x45 / M5, kit 4 Pcs.) <i>Replaces 1496030 (4x)</i>	1
15	1579301	(Hex socket head shoulder bolts, Black, Ø8x25 / M6, kit 4 Pcs.) <i>Replaces ZM008252A (4x)</i>	1
16	1579284	(Hex socket set screws with flat point, Black, M3x3, kit 25 Pcs.) <i>Replaces 116.1001 (25x)</i>	1
17	1579285	(Hex socket set screws with flat point, Black, M4x12, kit 25 Pcs.) <i>Replaces 116.1046 (25x)</i>	1
18	1579286	(Hex socket set screws with flat point, zinc plated blue, M5x5, kit 25 Pcs.) <i>Replaces 116.1081 (25x)</i>	1

(Hex-head screws, diverse, ...)

19	1579287	(Hex socket set screws with flat point, Black, M6x5, kit 25 Pcs.) <i>Replaces 1437272 (25x)</i>	1
20	1579288	(Hex socket set screws with flat point, zinc plated blue, M6x18, kit 25 Pcs.) <i>Replaces 1559106 (25x)</i>	1
21	1579289	(Hex socket set screws with flat point, Black, M6x35, kit 25 Pcs.) <i>Replaces 116.1133 (25x)</i>	1
22	1579290	(Hex socket set screws with flat point, Black, TufLok, M4x4, kit 10 Pcs.) <i>Replaces 1493792 (10x)</i>	1
23	1579291	(Hex socket set screws with flat point, Z.Plat.Bl., TufLok, M6x12, kit 10Pcs.) <i>Replaces 1439057 (10x)</i>	1
24	1579292	(Hex socket set screws with cone point, Black, M6x5, kit 25 Pcs.) <i>Replaces 116.2121 (25x)</i>	1
25	1579293	(Hex socket set screws with cone point, Black, M6x8, kit 25 Pcs.) <i>Replaces 116.2123 (25x)</i>	1
26	1579294	(Screws with cup point, Torx, zinc plated blue, M6x4, kit 25 Pcs.) <i>Replaces 1567763 (25x)</i>	1

(Washers)

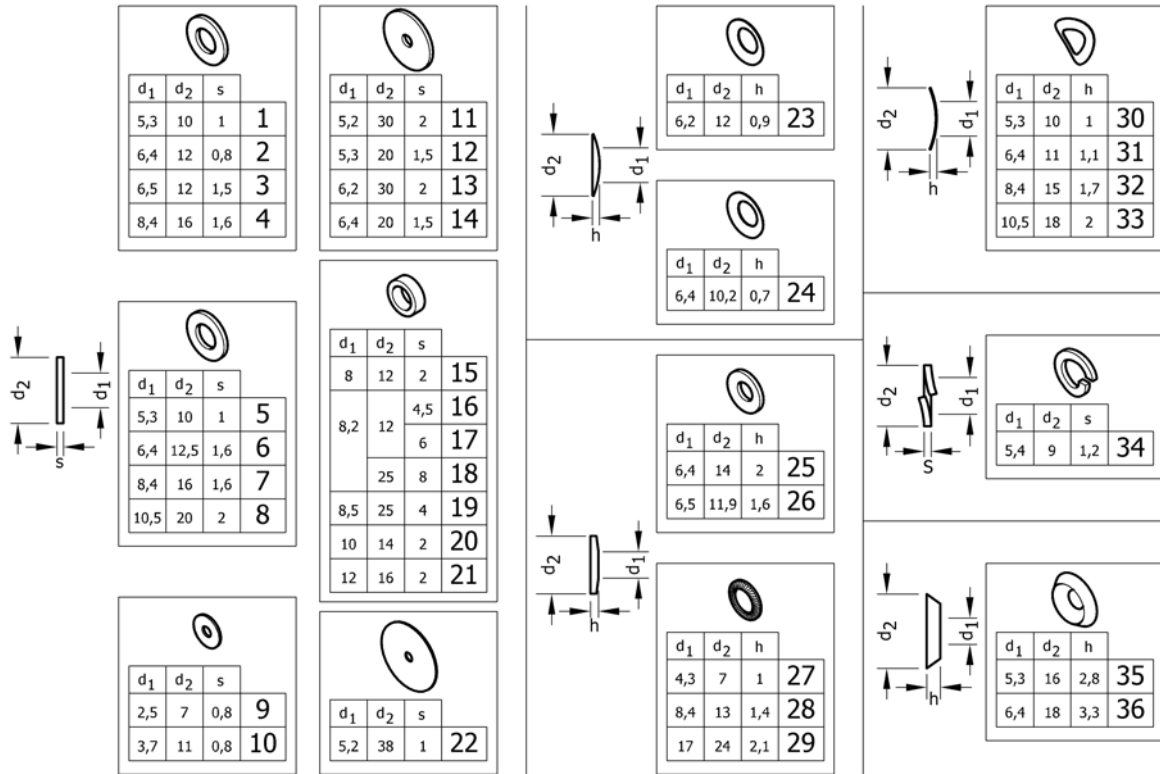


1	1579225	(Flat washers without chamfer, black, 5,3x10x1, kit 50 Pcs.) <i>Replaces 150.1004 (50x)</i>	1
2	1579226	(Flat washers without chamfer, black, 6,4x12x0,8, kit 50 Pcs.) <i>Replaces 150.2001 (50x)</i>	1
3	1579227	(Flat washers without chamfer, zinc plated blue, 6,5x12x1,5, kit 50 Pcs.) <i>Replaces 5729091 (50x)</i>	1
4	1579228	(Flat washers without chamfer, black, 8,4x16x1,6, kit 50 Pcs.) <i>Replaces 150.1012 (50x)</i>	1
5	1579229	(Flat washers without chamfer, polyamide, 5,3x10x1, kit 50 Pcs.) <i>Replaces 150.3003 (50x)</i>	1
6	1579230	(Flat washers without chamfer, polyamide, 6,4x12,5x1,6, kit 50 Pcs.) <i>Replaces 150.3004 (50x)</i>	1
7	1579231	(Flat washers without chamfer, polyamide, 8,4x16x1,6, kit 50 Pcs.) <i>Replaces 150.3005 (50x)</i>	1
8	1579232	(Flat washers without chamfer, polyamide, 10,5x20x2, kit 50 Pcs.) <i>Replaces 150.3006 (50x)</i>	1
9	1579233	(Flat washers without chamfer, zinc plated blue, 2,5x7x0,8, kit 50 Pcs.) <i>Replaces 150.2012 (50x)</i>	1
10	1579234	(Flat washers without chamfer, zinc plated blue, 3,7x11x0,8, kit 50 Pcs.) <i>Replaces ZM203501A (50x)</i>	1
11	1579235	(Flat washers without chamfer, black, 5,2x30x2, kit 20 Pcs.) <i>Replaces 150.2006 (20x)</i>	1
12	1579236	(Flat washers without chamfer, black, 5,3x20x1,5, kit 50 Pcs.) <i>Replaces 150.2010 (50x)</i>	1
13	1579237	(Flat washers without chamfer, black, 6,2x30x2, kit 20 Pcs.) <i>Replaces 150.2009 (20x)</i>	1
14	1579238	(Flat washers without chamfer, black, 6,4x25x1,5, kit 50 Pcs.) <i>Replaces 1451753 (50x)</i>	1
15	1579239	(Hinge rings, black, 8x12x2, kit 50 Pcs.) <i>Replaces 150.5002 (50x)</i>	1
16	1579240	(Distance sleeve, alu, 8,2x12x4,5, kit 20 Pcs.) <i>Replaces 1420970 (20x)</i>	1
17	1579241	(Distance sleeve, alu, 8,2x12x6, kit 20 Pcs.) <i>Replaces 1420969 (20x)</i>	1
18	1581292	(Distance sleeve, alu, 8,2x25x8, kit 2 Pcs.) <i>Replaces 151.1002 (2x)</i>	1

(Washers)

19	1583052	(Distance sleeve, alu, black, 8,5x25x4, kit 10 Pcs.) <i>Replaces 151.1014 (10x)</i>	1
20	1579242	(Hinge rings, zinc plated blue, 10x14x2, kit 50 Pcs.) <i>Replaces 1506666 (50x)</i>	1
21	1579243	(Hinge rings, black, 12x16x2, kit 50 Pcs.) <i>Replaces 150.5006 (50x)</i>	1
22	1579244	(Rubber washers, 5,2x38x1, kit 20 Pcs.) <i>Replaces 1424891 (20x)</i>	1
23	1579245	(Disc springs, black, 6,2x12x0,9, kit 50 Pcs.) <i>Replaces 160.1015 (50x)</i>	1
24	1579246	(Conical spring washers, zinc plated blue, 6,4x10,2x0,7, kit 50 Pcs.) <i>Replaces 161.1004 (50x)</i>	1
25	1579247	(Conical spring washers, black, 6,4x14x2, kit 50 Pcs.) <i>Replaces 161.2004 (50x)</i>	1
26	1579248	(Conical spring washers, zinc plated blue, 6,5x11,9x1,6, kit 50 Pcs.) <i>Replaces ZM206001A (50x)</i>	1
27	1579249	(Ribbed lock washers, black, 4,3x7x1, kit 50 Pcs.) <i>Replaces 155.2002 (50x)</i>	1
28	1579250	(Ribbed lock washers, zinc plated blue, 8,4x13x1,4, kit 50 Pcs.) <i>Replaces 155.2005 (50x)</i>	1
29	1579251	(Ribbed lock washers, zinc plated blue, 17x24x2,1, kit 50 Pcs.) <i>Replaces 155.2009 (50x)</i>	1
30	1579252	(Curved spring washers, black, 5,3x10x1, kit 50 Pcs.) <i>Replaces 1438515 (50x)</i>	1
31	1579253	(Curved spring washers, black, 6,4x11x1,1, kit 50 Pcs.) <i>Replaces 1438516 (50x)</i>	1
32	1579254	(Curved spring washers, zinc plated blue, 8,4x15x1,7, kit 50 Pcs.) <i>Replaces ZM208003A (50x)</i>	1
33	1579255	(Curved spring washers, zinc plated blue, 10,5x18x2, kit 50 Pcs.) <i>Replaces ZM201581A (50x)</i>	1
34	1579256	(Split spring lock washers, black, 5,4x9x1,2, kit 50 Pcs.) <i>Replaces 155.1003 (50x)</i>	1
35	1579257	SKIVE FORSENKET 90G M5° 3x16x2,8 50 stk <i>Replaces 150.6003 (50x)</i>	1
36	1579258	SKIVE FORSENKET 90G M6° 4x18x3,3 50 stk <i>Replaces 150.6004 (50x)</i>	1

(Nuts)

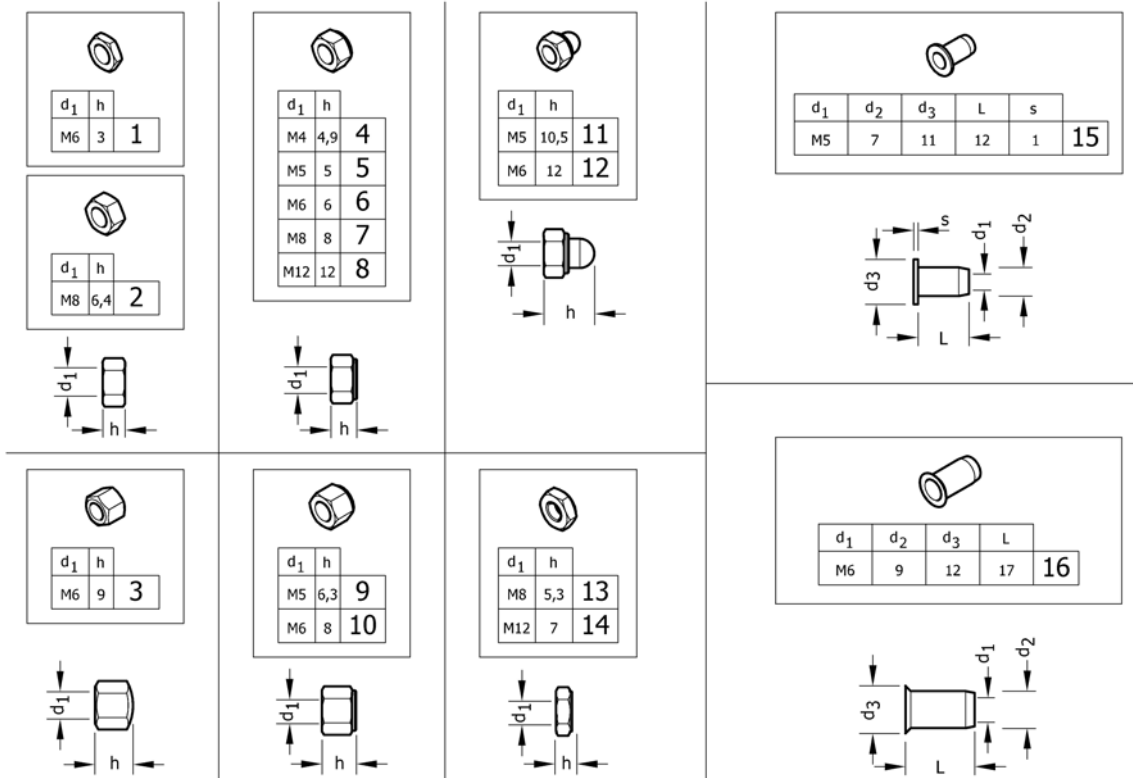


1	1579207	(Hex jam nuts, black, M6, kit 25 Pcs.) <i>Replaces 108.3004 (25x)</i>	1
2	1579208	(Hexagon nuts , black, M8, kit 25 Pcs.) <i>Replaces 1420997 (25x)</i>	1
3	1579209	(Hex cap nuts, low type, black, M6, kit 25 Pcs.) <i>Replaces 109.5009 (25x)</i>	1
4	1579210	(Prev.Torque lock nuts, thin type, with polyam.Insert, black, M4, kit 25 Pcs.) <i>Replaces 109.1002 (25x)</i>	1
5	1579211	(Prev.Torque lock nuts, thin type, with polyam.Insert, black, M5, kit 25 Pcs.) <i>Replaces 109.1003 (25x)</i>	1
5	1579212	(Prev.Torque lock nuts, thin type, with polyam.Insert, zi.Bl., M5, kit 25 Pcs.) <i>Replaces 109.5008 (25x)</i>	1
6	1579213	(Prev.Torque lock nuts, thin type, with polyam.Insert, black, M6, kit 25 Pcs.) <i>Replaces 109.1004 (25x)</i>	1
6	1579214	(Prev.Torque lock nuts, thin type, with polyam.Insert, zi.Bl., M6, kit 25 Pcs.) <i>Replaces ZM106001A (25x)</i>	1
7	1579215	(Prev.Torque lock nuts, thin type, with polyam.Insert, black, M8, kit 25 Pcs.) <i>Replaces 109.1005 (25x)</i>	1
8	1579216	(Prev.Torque lock nuts, thin type, with polyam.Insert, black, M12, kit 4 Pcs.) <i>Replaces 109.1007 (4x)</i>	1
9	1579217	(Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M5, kit 25 Pcs.) <i>Replaces 1493771 (25x)</i>	1
10	1579218	(Preva.Torq.Hex lock nuts, high type with poly.Insert, black, M6, kit 25 Pcs.) <i>Replaces 109.5001 (25x)</i>	1
11	1579219	(Prevail.Torque hex domed cap nuts with polyam.Insert, black, M5, kit 25 Pcs.) <i>Replaces 109.5006 (25x)</i>	1
12	1579220	(Prevail.Torque hex domed cap nuts with polyam.Insert, black, M6, kit 25 Pcs.) <i>Replaces 109.5003 (25x)</i>	1
13	1579221	(Prevailing torque hex lock nuts, all-metal, thin typeblack, M8, kit 10 Pcs.) <i>Replaces 109.5005 (10x)</i>	1

(Nuts)

14	1579222	(Prevailing torque hex lock nuts, all-metal, thin typeblack, M12, kit 4 Pcs.) <i>Replaces 109.5004 (4x)</i>	1
15	1579223	(Blind rivet nuts, flat head, open type, black, M5, kit 10 Pcs.) <i>Replaces 180.4007 (10x)</i>	1
16	1579224	(Blind rivet nut with low countersunk head, open type, black, M6, kit 10 Pcs.) <i>Replaces 180.4105 (10x)</i>	1

(Pins, rivets, ...)

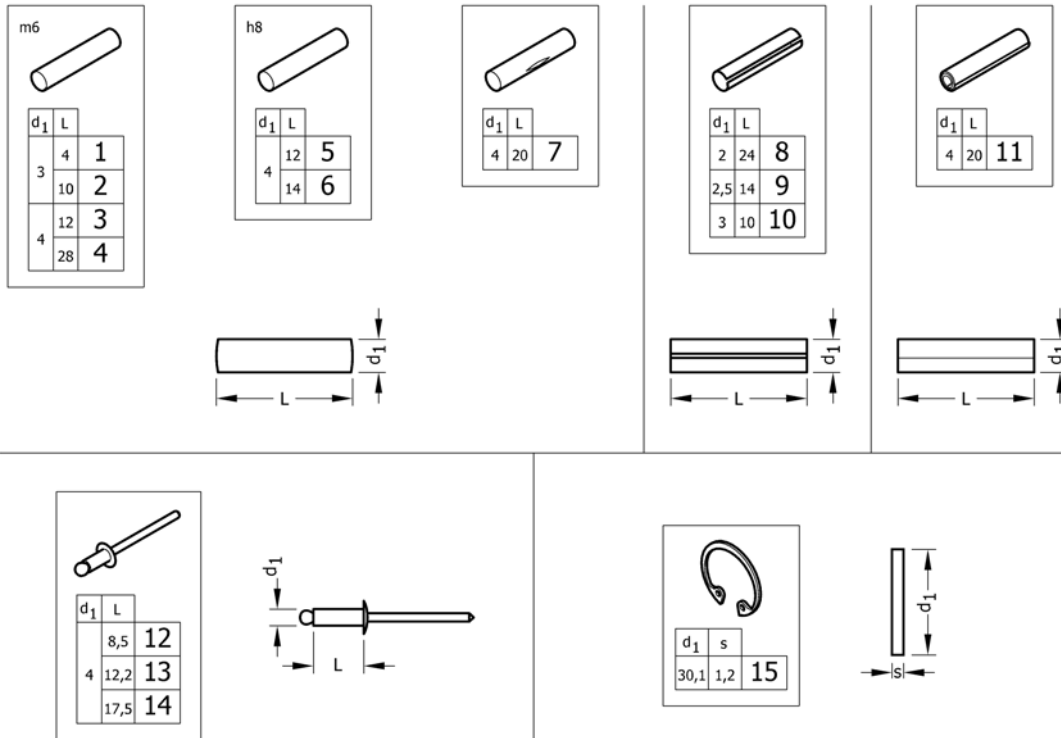

1	1579302	(Parallel pins, Ø3x4 m6, kit 10 Pcs.) <i>Replaces 1439336 (10x)</i>	1
2	1579303	(Parallel pins, Ø3x10 m6, kit 10 Pcs.) <i>Replaces 1439337 (10x)</i>	1
3	1579304	(Parallel pins, Ø4x12 m6, kit 20 Pcs.) <i>Replaces 200.1124 (20x)</i>	1
4	1579305	(Parallel pins, Ø4x28 m6, kit 20 Pcs.) <i>Replaces 1437344 (20x)</i>	1
5	1579306	(Parallel pins, Ø4x12 h8, kit 20 Pcs.) <i>Replaces 1487663 (20x)</i>	1
6	1579307	(Parallel pins, Ø4x14 h8, kit 20 Pcs.) <i>Replaces 1487662 (20x)</i>	1
7	1579308	(Grooved pins, center grooved, Ø4x20, kit 20 Pcs.) <i>Replaces 1517718 (20x)</i>	1
8	1579309	(Slotted spring pins, heavy duty, Black, Ø2x24, kit 20 Pcs.) <i>Replaces 200.4011 (20x)</i>	1
9	1579310	(Slotted spring pins, heavy duty, Black, Ø2,5x14, kit 20 Pcs.) <i>Replaces 200.4007 (20x)</i>	1
10	1579311	(Slotted spring pins, heavy duty, stainless steel, Ø3x10, kit 20 Pcs.) <i>Replaces 1567997 (20x)</i>	1
11	1579312	(Spiral spring pins, Black, Ø4x20, kit 20 Pcs.) <i>Replaces 200.3167 (20x)</i>	1
12	1579313	(Dome head blind rivets, Ø4x8,5, kit 20 Pcs.) <i>Replaces 1439292 (20x)</i>	1
13	1579314	(Dome head blind rivets, Ø4x12,2, kit 20 Pcs.) <i>Replaces 180.5116 (20x)</i>	1
14	1579315	(Dome head blind rivets, Ø4x17,5, kit 20 Pcs.) <i>Replaces 180.5019 (20x)</i>	1
15	1579316	(Retaining rings for bores, Std.Design, Z.plat.Blue, Ø30,1X1,2, kit 20 Pcs.) <i>Replaces ZM50J281A (20x)</i>	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk