# Stand Assist

en	Sling	User Manual	2
da	Sejl	Brugsanvisning	3
de	Patientengurte	Gebrauchsanweisung	4
es	Eslinga	Manual del usuario	6
fi	Nostoliina	Käyttöohje	7
fr	Sangles	Manuel d'utilisation	8
it	Imbracatura	Manuale d'uso	10
nl	Liftmat	Gebruiksaanwijzing	11
no	Seil	Bruksanvisning	13
pt	Cesta de transferência	Manual de utilização	14

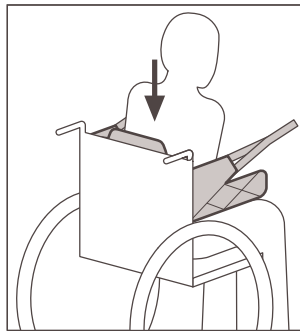
Figure 1

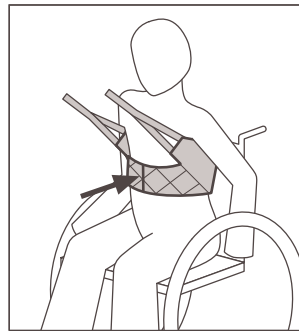
Figure 2

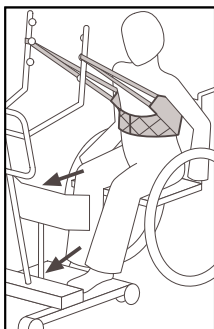
Figure 3

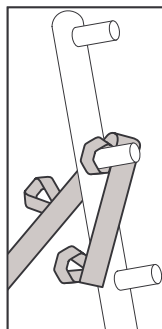
Figure 4

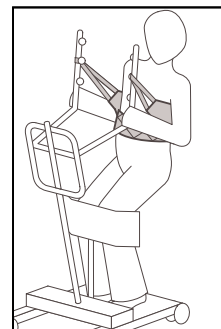
Figure 5

This manual **MUST** be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

## General

### Symbols

	Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words. <b>WARNING!</b> Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.  <b>CAUTION!</b> Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.
	Gives useful tips, recommendations and information for efficient, trouble-free use.
	<b>IMPORTANT</b> indicates a hazardous situation that could result in damage to property if it is not avoided.
	This product complies with Directive 93/42/EEC concerning medical devices.  The launch date of this product is stated in the CE declaration of conformity.
	Safe Working Load
	Consult the user manual.
	DO NOT exceed 95°C washing temperature.
	DO NOT bleach.
	Tumble Dry, Normal, Low Heat
	DO NOT iron.

### Intended Use

The Invacare Stand Assist sling used in conjunction with a standing hoist can provide an excellent transfer solution for those that have the ability to weight bear but are unable to independently stand. It also promotes independence and dignity of the client and allows a simple procedure for the caregiver.

The Invacare Stand Assist sling is suitable for clients with a significant trunk and head control and with an adequate arm and hand dexterity to hold the necessary bar of the hoist to stabilise themselves during the lift. The weight bearing ability should be bilateral.

### Service Life

All Invacare slings are guaranteed for one year. Expected service life of the product is 1-5 years. Service life time varies depending on the fabric, frequency of use, laundering practices, and weight carried.

## Safety

### General Guidelines

#### WARNING!

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Please check your sling on delivery. If the sling is not in perfect condition, please contact your supplier.

### Safe Working Load

	Solid	
<b>S</b>	1484685	200 kg
<b>M</b>	1484686	200 kg
<b>L</b>	1484687	200 kg
<b>XL</b>	1484688	200 kg

## Usage

### Sling application

#### WARNING!

- Slings should be checked regularly for wear and damage (in accordance with each country's regulations). NEVER use a damaged sling.
- If you are in any doubt, please do not attempt to use the sling and seek the advice of a professional or your supplier.
- Invacare recommends that a risk assessment is always carried out by a professional prior to issuing lifting equipment. Always choose the sling design and size according to the client's weight, size and physical ability.
- If you are in doubt about the use of Invacare's slings with other manufacturers' equipment, contact Invacare or your supplier for advice.

### Sling application — Sitting in chair or wheelchair

1. Lean the client forward ensuring that he is well supported to reduce risk of falling and to promote feelings of security.
2. Slide the sling down behind the back of the client in a central position with the aperture apex aligned with the base of the spine.
3. Sit the client back against the sling. **Figure 1.**
4. Fasten the vest round the clients waist by using the adjustable buckle and ensure that the clients arms are outside the Invacare Stand Assist sling. **Figure 2.**
5. Place the clients knees secure against the kneepad and make sure that the feet are positioned well on the footplate. **Figure 3.**

## Attach loops to the bar

### WARNING!

- Use only CE marked spreader bars and hoists, preferably with a rated Safe Working Load equal to or higher than that of the sling. If the SWL is lower than that of the sling, the SWL of the spreader bar or hoist is applicable.
- Always check that all the straps are properly attached to the spreader bar or lifting strap before attempting to lift the client.

1. Determine the best attachment point for the Invacare Stand Assist sling according to the height of the client. For a tall client the attachment point should be closest to the client and visa verse.

The colour coded straps can be used to place the client in various positions. The colour makes it easy to connect both sides equally.

2. Attach the sling to the attachment point from the outside of the stand assist. **Figure 4.**
3. Make sure the following:
  - The sling is wrapped around the outside of each lift arm.
  - Use the closest colour coded strap to the client that maintains stability and comfort.
  - The colour coded strap sits flush against the attachment knob.
  - The same colour coded point is used on each lift arm.
  - Always use the same colour code on each side of the Invacare Stand Assist sling.
4. Before lifting the client always instruct the client to do the following:
  - Hold on the handgrips on both sides of the stand assist.
  - Lean back into the Invacare Stand Assist sling. **Figure 5.**

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## Algemeen

### Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

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- Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

#### LET OP!

- Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

#### IMPORTANT

- Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

Date of manufacture.

This product complies with Directive 93/42/EEC concerning medical devices.

The launch date of this product is stated in the EC declaration of conformity.

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©2017 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB