

Spare Parts Catalogue

Version: 2015/08/21c EN



Invacare® **TDX-SP**



Yes, you can.®

General Information

For any order of spare parts the following information is required:

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.

The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.



Yes, you can.®

TABLE OF CONTENTS

Information & User Manuals

User Manuals TDX-SP	9
User Manuals REM550	10
User Manuals REM24SD	11
User Manuals Shark2	12
User Manuals REM A & REM B	13
User Manuals G90	14
User Manuals G91	15
User Manuals IDC Dual Control	16
User Manuals REM34 & REM41 Tray/Centre Control	17
User Manuals ACS Handprogrammer	18
User Manual Proximity Head Remote DSL-105	19
User Manuals Ultra Low Maxx Seat	20

Chassis

Walking Beam ==> 06.09.11	21
Walking Beam SSD-Motors	22
Rear Pivot Assembly	23
Rear Pivot Assembly (Mounting)	24
Cylinders, Locking Gas	25
Fender	26
Shrouds and Hardware	27
Battery Box Hardware	28

Seat

Seat	29
Standard Seat	30
Powered Tilt 50° & manuell	31
Tiltmodule 28°	32
Lifter (incl. 28°-Tilt)	33
Bellow	34
Seat frames for Balle modules	35
Adapter sheets	36
Seatplates	37
Seat Cushion for Standard Seat	38
Contour Seat	39
Powered Tilt 50° & manuell	31
Tiltmodule 28°	32
Lifter (incl. 28°-Tilt)	33
Bellow	34
Seat frames for Balle modules	35
Adapter sheets	36
Seatplates	37
Seat Cushion for Contour Seat	40
Flex 2 Seat	41
Powered Tilt 50° & manuell	31
Tiltmodule 28°	32
Lifter (incl. 28°-Tilt)	33
Bellow	34
Seat frames for Balle modules	35



Yes, you can.®

TABLE OF CONTENTS

Adapter sheets	36
Seatplates	37
Seat Cushion for Flex Seat	42
Junior Seat	43
Powered Tilt 50° & manuell	31
Tiltmodule 28°	32
Lifter (incl. 28°-Tilt)	33
Bellow	34
Seat frames for Balle modules	35
Adapter sheets	36
Seatplates	37
Seat Cushion for Junior Seat	44
Modulite Seat	45
man. settable Tilt	46
powered Tilt 0°-30°	47
Lifter	48
One Piece Seat Plate	49
Telescopic Seat Frame and Plate	50
Telescopic Seat Frame for Tilt 12° / 20°	51
Telescopic Seat Frame for Tilt 30° / Lifter	52
Armrest bracket	53
Legrest Support	54
Seat Plate	55
Telescopic Seat Frame and Sling Seat	56
Telescopic Seat Frame for Tilt 12° / 20°	57
Telescopic Seat Frame for Tilt 30° / Lifter	58
Armrest bracket	53
Legrest Support	59
Sling Seat	60
Telescopic Seat Frame and Universal Adapter	61
Telescopic Seat Frame for Tilt 12° / 20°	51
Telescopic Seat Frame for Tilt 30° / Lifter	52
Armrest bracket	53
Legrest Support	54
Adapter Kit for Universal Seat	62
Seat Cushion	63

Backrest

Backrest	66
Backrest for Standard Seat	67
Backrest Standard Seat, settable	68
Backrest Recliner	69
Backrest for Contour Seat	70
Backrest Flex Seat	71
Backrest Adjustment Flex/Contour manual	72
Backrest adjustment Flex/Contour electric	73
Backrest Cushion for Contour Seat	75
Backrest for Flex Seat	76
Backrest Flex Seat	71
Backrest Adjustment Flex/Contour manual	72
Backrest adjustment Flex/Contour electric	73
Backrest Cushion for Flex Seat	77



Yes, you can.®

TABLE OF CONTENTS

Backrest for Junior Seat	78
Backrest / Junior Seat	79
Backrest Cushion for Junior Seat	80
Backrest for Modulite Seat	81
Support for Backrest Frame	82
Backrest Frame	83
Sling Back, tension adjustable	84
Backrest support	85
Backposts	86
Connection tube	87
Back Recliner / Stabilization bracket	88
Backrest Cushions	89
Flex 3 / Laguna	90
Matrx Posture Back	92
Matrx Elite T/R Back	93
Matrx Elite Deep Back	94
Matrx Elite (Full) Back	95

Legrests / Hangers

Legrest fixation	96
Hanger Vari A (04/2009 =>)	97
Hanger Vari F	98
Hangers ADM LH/RH manual	99
Hanger Vari ADE/ADP electric (07/2005 =>)	100
Central Legrest, manual - Standard- / Easy Adapt-Seat	101
Central Legrest, manual - Modulite Seat	102
Central Legrest LNX electr. - Standard- / Easy Adapt-Seat	103
Central Legrest LNX electr. - Modulite Seat	104
Centre-mounted Legrests	105
Bracket Centr. Legrest LNX - Narrow Base	106

Footplates / Support tubes

Footrests Vari, Flip-Up (01.08.10 =>)	107
Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)	108
Footplate	109
Footplate with support tube fix RH/LH	110
Footplate with support tube angle adjust. RH/LH	111
Support Tube with Footplate for central Legrest	112
Footrest Lower Parts	113

Armrests

Armrests	114
Armrest for Standard Seat	115
Armrest Standard	116
Armrests cpl.	117
T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>	118
Armrest for Contour Seat	119
Armrest Standard	116
Armrests cpl.	117
Mounting bracket Flip-up armrests	120
Following armrests	121



Yes, you can.®

TABLE OF CONTENTS

Sliding Armrest	122
Skirtguard for following armrest	123
Armrest for Flex Seat	124
Armrest Standard	116
Armrests cpl.	117
Mounting bracket Flip-up armrests	120
Following armrests	121
Sliding Armrest	122
Skirtguard for following armrest	123
Armrest for Junior Seat	125
Armrests cpl.	117
Armrests for Modulite Seat	126
T-Armrest	127
Padded Armrest	128
Inrigged Armrest	129
Flip-Up Armrest	130
Following Armrest	131
Hip Support	132

Castors

Castors Front/Rear	133
--------------------------	-----

Drive Wheels

Drive Wheels	134
--------------------	-----

Propulsion

Motor & Gearbox ==> 06.09.11	135
Overview Propulsion SSD	136

Electronic

Electronic	139
ACS Electronic	140
ACS Electronic Modules	141
Powermodule ACS2	142
ACT2/4 for ACS2	143
Bracket ACT narrow base	144
Lightboard ACS2	145
G-Trac-sensor	146
Holder for G-Trac	147
ACS Remote & Brackets	148
REM 24 SD with brackets	149
Remote REM 24SD	150
Remote Bracket Standard	151
Remote Bracket Swing away & height adjust.	152
Remote Bracket Attendant	153
REM A & B with brackets	154
Remote Bracket Standard	151
Remote Bracket Swing away & height adjust.	155
Remote Bracket Attendant	153
REM 550 with brackets	156



Yes, you can.®

TABLE OF CONTENTS

Remote Bracket Standard	151
Remote Bracket Swing away & height adjust.	152
Remote Bracket Attendant	153
REM 34 with brackets	157
Remote Bracket Standard	158
Remote Bracket Swing away & height adjust.	159
G90A with brackets	160
Remote Bracket Standard	161
Remote Bracket Swing away & height adjust.	162
Remote Bracket Attendant	163
G91S with brackets	164
Remote Bracket Standard	165
Centre control REM 41	166
Small Chin Control with brackets	167
Small Chincontrol	168
Framework for chincontrol	169
Remote Dual Control with Brackets	170
Remote Bracket Dual Control for Standard-/Contour-/Flex-/Junior-Seat	171
Remote Bracket Dual Control for Modulite Seat	172
Intuitive Dual Control with brackets	173
ACS Cables & Accessories	174
Bus-Cables ACS	175
T-Adapter	176
T-Adapter - Modulite Seat	177
Programming Kits	178
Magnetkey for Remotes	179
Shark Electronic	180
Shark Electronic Modules	181
Shark Powermodule	182
Lightboard ACS2	145
Shark Remote & Brackets	183
Remote Shark with Brackets	184
Remote Bracket Standard	151
Remote Bracket Swing away & height adjust.	155
Remote Dual Control with Brackets	185
Remote Bracket Dual Control	186
Shark Cables & Accessories	187
Bus-Cables Shark	188
Cables Shark	189
Programming Kits	178

Cable, Charger & Batteries

Chargers	190
Batteries	191
Battery support	192
Battery Tray Assembly and Wire Harnesses	193

Safety Options & Accessories

Light System	194
Rear Light Protector Kit	195
Lightsystem LED Front	196



Yes, you can.®

TABLE OF CONTENTS

Reflective Sign	197
Lap Belt	198
Lap Belts (Modulite Seat)	199
H-Belts (Modulite Seat)	200
Rearview Mirror	201

Seat Accessories

Backrest Extension Flex	202
Abduction Wedge	203
Lateral Support - Flex 2 Seat	204
Lateral Support (Dartex)	205
Lateral Support, extended arm (Dartex)	206
Lateral Support - Modulite Seat	207
Lateral Support (Dartex)	208
Lateral Support, extended arm (Dartex)	206
Lateral Support - Junior Seat	209
Head- & Neckrests	210
Push Handle	211

Accessories

Caneholder	212
Caneholder for Standard Seat	213
Caneholder for Contour/Flex Seat	214
Cane holder (Modulite: backrest frame)	215
Cane holder (Modulite: back posts man recliner)	216
Cane holder (Modulite: back posts pow recliner)	217
Trays	218
Hour Counter	220
KLICKfix® bag	221
KLICKfix® bracket for Fixed Armrest	222
KLICKfix® bracket for T-Armrest	223
KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite)	224
Bag	225
Bumper/Sport Accessories	226
T-Nuts for Accessories Modulite Seat	227

User Manuals TDX-SP

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	CS 1531137	User Manual TDX-SP Czech		1
	DA 1537969	User Manual TDX-SP Dansk		1
	DE 1521463	User Manual TDX-SP German		1
	EN 1521464	User Manual TDX-SP English		1
	ES 1522212	User Manual TDX-SP Spanish		1
	FR 1522172	User Manual TDX-SP French		1
	IT 1522187	User Manual TDX-SP Italian		1
	LT 1531140	User Manual TDX-SP Lithuanian		1
	NO 1537970	User Manual TDX-SP Norwegian		1
	PL 1531138	User Manual TDX-SP Polish		1
	RU SP1549025	User Manual TDX-SP Russian		1
	SL 1531139	User Manual TDX-SP Slovenian		1
	SV 1522687	User Manual TDX-SP Swedish		1

User Manuals REM550

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DA 1528088	User Manual REM550 Danish		1
	DE 1528089	User Manual REM550 German		1
	EN 1528090	User Manual REM550 English		1
	ES 1528091	User Manual REM550 Spanish		1
	FI 1528092	User Manual REM550 Finsk		1
	FR 1528093	User Manual REM550 French		1
	IT 1528094	User Manual REM550 Italiano		1
	NL 1528095	User Manual REM550 Dutch		1
	NO 1528096	User Manual REM550 Norsk		1
	PT 1528097	User Manual REM550 Portuguese		1
	SV 1528098	User Manual REM550 Swedish		1
	HR SP1533279	User Manual REM550 Croatian		1
	CS SP1533280	User Manual REM551 Czech		1
	ET SP1533281	User Manual REM552 Estonian		1
	HU SP1533282	User Manual REM553 Hungarian		1
	LT SP1533283	User Manual REM554 Lithuanian		1
	PL SP1533284	User Manual REM555 Polish		1
	SL SP1533285	User Manual REM556 Slovenian		1

User Manuals REM24SD

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1436142	User Manual REM24SD German		1
	FR 1436143	User Manual REM24SD French		1
	EN 1436144	User Manual REM24SD English		1
	SV 1436145	User Manual REM24SD Swedish		1
	DA 1436146	User Manual REM24SD Danish		1
	NO 1436147	User Manual REM24SD Norwegian		1
	NL 1436148	User Manual REM24SD Dutch		1
	ES 1436149	User Manual REM24SD Spanish		1
	PT 1436150	User Manual REM24SD Portuguese		1
	IT 1436385	User Manual REM24SD Italian		1
	FI 1436386	User Manual REM24SD Finnish		1
	HR SP1533251	User Manual REM24SD Croatian		1
	CS SP1533252	User Manual REM24SD Czech		1
	ET SP1533253	User Manual REM24SD Estonian		1
	HU SP1533254	User Manual REM24SD Hungarian		1
	LT SP1533255	User Manual REM24SD Lithuanian		1
	PL SP1533256	User Manual REM24SD Polish		1
	SL SP1533257	User Manual REM24SD Slovenian		1
	LV SP1543152	User Manual REM24SD Latvian		1

User Manuals Shark2

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1451346	User Manual Shark II German		1
	FR 1451347	User Manual Shark II French		1
	SV 1451348	User Manual Shark II Swedish		1
	DA 1451349	User Manual Shark II Danish		1
	NO 1451350	User Manual Shark II Norwegian		1
	NL 1451351	User Manual Shark II Dutch		1
	ES 1451352	User Manual Shark II Spanish		1
	PT 1451353	User Manual Shark II Portuguese		1
	IT 1451354	User Manual Shark II Italian		1
	FI 1451355	User Manual Shark II Finnish		1
	EN 1451356	User Manual Shark II English		1
	SL 1451584	User Manual Shark II Slovenian		1
	CS SP1533297	User Manual Shark2 CS		1
	ET SP1533298	User Manual Shark2 EE		1
	HU SP1533299	User Manual Shark2 HU		1
	LT SP1533300	User Manual Shark2 LT		1
	PL SP1533301	User Manual Shark2 PL		1
	BG SP1546061	User Manual Shark II Bulgarian		1
	RU SP1570186	User Manual Shark II Russian		1
	HR SP1574028	User Manual Shark II Croatian		1
	LV SP1576075	User Manual Shark II Latvian		1

User Manuals REM A & REM B

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DA 1529171	User Manual REM A & B Dansk		1
	DE 1529172	User Manual REM A & B German		1
	EN 1529173	User Manual REM A & B English		1
	ES 1529174	User Manual REM A & B Spanish		1
	FI 1529228	User Manual REM A & B Finsk		1
	FR 1529229	User Manual REM A & B French		1
	IT 1529230	User Manual REM A & B Italian		1
	NL 1529231	User Manual REM A & B Dutch		1
	NO 1529232	User Manual REM A & B Norsk		1
	PT 1529233	User Manual REM A & B Portuguese		1
	SV 1529234	User Manual REM A & B Swedish		1
	HR SP1533258	User Manual REM A & B Croatian		1
	CS SP1533259	User Manual REM A & B Czech		1
	ET SP1533265	User Manual REM A & B Estonian		1
	HU SP1533266	User Manual REM A & B Hungarian		1
	LT SP1533267	User Manual REM A & B Lithuanian		1
	PL SP1533268	User Manual REM A & B Polish		1
	SL SP1533269	User Manual REM A & B Slovenian		1
	RO SP1533270	User Manual REM A & B Romanian		1

User Manuals G90

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	1523674	User Manual G90 English		1

User Manuals G91

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1536441	User Manual G91 German		1
	DA 1537929	User Manual G91 Danish		1
	NL 1537930	User Manual G91 Dutch		1
	EN 1537931	User Manual G91 English		1
	FI 1537932	User Manual G91 Finnish		1
	FR 1537933	User Manual G91 French		1
	IT 1537934	User Manual G91 Italian		1
	NO 1537935	User Manual G91 Norwegian		1
	PT 1537936	User Manual G91 Portuguese		1
	ES 1537937	User Manual G91 Spanish		1
	SV 1537938	User Manual G91 Swedish		1

User Manuals IDC Dual Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1521480	User Manual Dual-Control German		1
	EN 1521481	User Manual Dual-Control English		1
	FR 1521482	User Manual Dual-Control French		1
	NL 1521483	User Manual Dual-Control Dutch		1
	NO 1521484	User Manual Dual-Control Norwegian		1
	SV 1521485	User Manual Dual-Control Swedish		1
	DA 1521486	User Manual Dual-Control Danish		1
	IT 1522252	User Manual Dual-Control Italian		1
	PT 1522254	User Manual Dual-Control Portuguese		1
	ES 1522255	User Manual Dual-Control Spanish		1

Invacare® TDX-SP, TDX-SP nB, TDX Sprint | Information & User Manuals
User Manuals REM34 & REM41 Tray/Centre Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1538022	User Manual REM41 Tray Control Danish		1
EN	SP1538023	User Manual REM41 Tray Control English		1
ES	SP1538024	User Manual REM41 Tray Control Spanish		1
FR	SP1538025	User Manual REM41 Tray Control French		1
IT	SP1538026	User Manual REM41 Tray Control Italian		1
NL	SP1538027	User Manual REM41 Tray Control Dutch		1
NO	SP1538028	User Manual REM41 Tray Control Norwegian		1
PT	SP1538029	User Manual REM41 Tray Control Portuguese		1
SV	SP1538030	User Manual REM41 Tray Control Swedish		1
CS	SP1538031	User Manual REM41 Tray Control Czech		1
SL	SP1538032	User Manual REM41 Tray Control Slovenian		1
DE	SP1541402	User Manual REM41 Tray Control German		1

User Manuals ACS Handprogrammer

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DE	1419316	User Manual Handheld Programmer German		1
EN	1419317	User Manual Handheld Programmer English		1

User Manual Proximity Head Remote DSL-105

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1555731	User Manual Proximity Head Remote DSL-105 English		1
DE	SP1555732	User Manual Proximity Head Remote DSL-105 German		1
FR	SP1555733	User Manual Proximity Head Remote DSL-105 French		1
DA	SP1555839	User Manual Proximity Head Remote DSL-105 Danish		1
ES	SP1555840	User Manual Proximity Head Remote DSL-105 Spanish		1
FI	SP1555841	User Manual Proximity Head Remote DSL-105 Finnish		1
IT	SP1555842	User Manual Proximity Head Remote DSL-105 Italian		1
NL	SP1555843	User Manual Proximity Head Remote DSL-105 Dutch		1
NO	SP1555844	User Manual Proximity Head Remote DSL-105 Norwegian		1
PT	SP1555845	User Manual Proximity Head Remote DSL-105 Portuguese		1
SV	SP1555846	User Manual Proximity Head Remote DSL-105 Swedish		1


User Manuals Ultra Low Maxx Seat

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		


Pos.	Part number	Description	Valid from → until	Qty
DA	SP1585726	User Manual Ultra Low Maxx Seat - Danish		1
DE	SP1585727	User Manual Ultra Low Maxx Seat - German		1
EN	SP1585725	User Manual Ultra Low Maxx Seat - English		1
FI	SP1585728	User Manual Ultra Low Maxx Seat - Finish		1
FR	SP1585729	User Manual Ultra Low Maxx Seat - French		1
IT	SP1588266	User Manual Ultra Low Maxx Seat - Italian		1
NL	SP1585730	User Manual Ultra Low Maxx Seat - Dutch		1
NO	SP1585731	User Manual Ultra Low Maxx Seat - Norwegian		1
PT	SP1585732	User Manual Ultra Low Maxx Seat - Portuguese		1
SV	SP1585733	User Manual Ultra Low Maxx Seat - Swedish		1

Walking Beam ==> 06.09.11

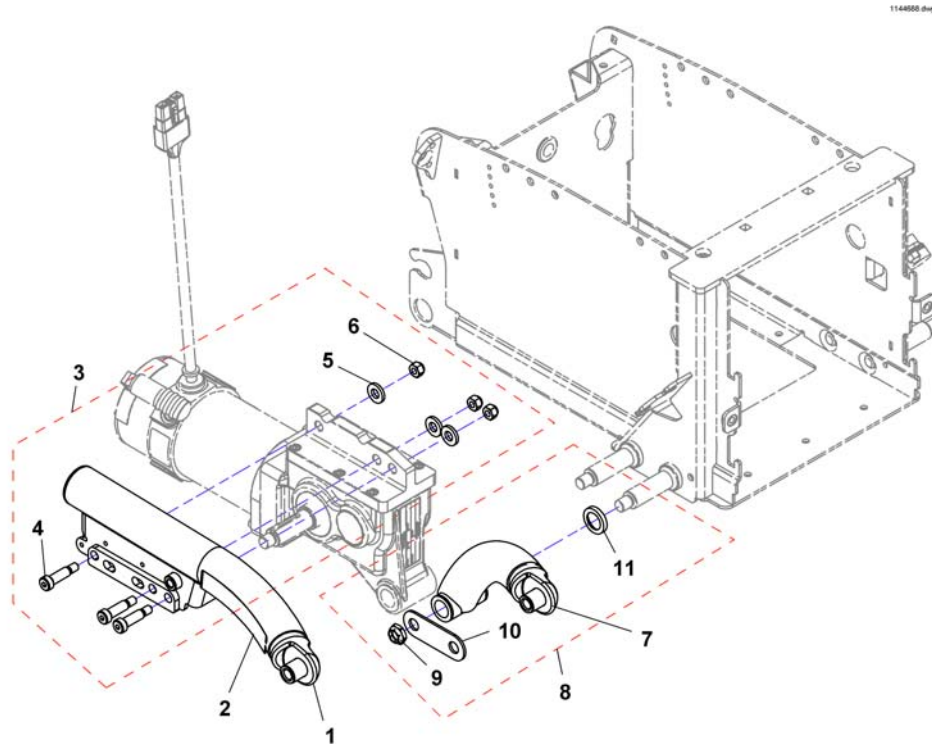


Pos.	Part number	Description	Valid from → until	Qty
1	SP1523256	Walking Beam Assembly (electric red) with TDX SP Decal, LH		1
1	SP1523255	Walking Beam Assembly (electric red) with TDX SP Decal, RH		1
1	SP1523258	Walking Beam Assembly (wet black) with TDX SP Decal, LH		1
1	SP1523257	Walking Beam Assembly (wet black) with TDX SP Decal, RH		1
2	SP1523167	Decal, TDX SP (Yellow)		1
2	SP1523012	Decal, TDX SP (Black)		1
3	SP1523178	Kit, Walking Beam Mounting Hardware		1
7	SP1523199	Link, Lower Pivot (wet black)-Left		1
7	SP1523198	Link, Lower Pivot (wet black)-Right		1
7	SP1563163	Link, Lower Pivot (electric red)-Left		1
7	SP1563169	Link, Lower Pivot (electric red)-Right		1
8	SP1523168	Kit, Lower Pivot Link / 4-Pole Motor Mounting Hardware		1
4	SP1522942	Screw, Socket Head Shoulder (3/8 x1")		1
5	SP1522966	Washer (7/16 x13/16 x1/8")		1
10	-	No longer available.		0

Walking Beam SSD-Motors

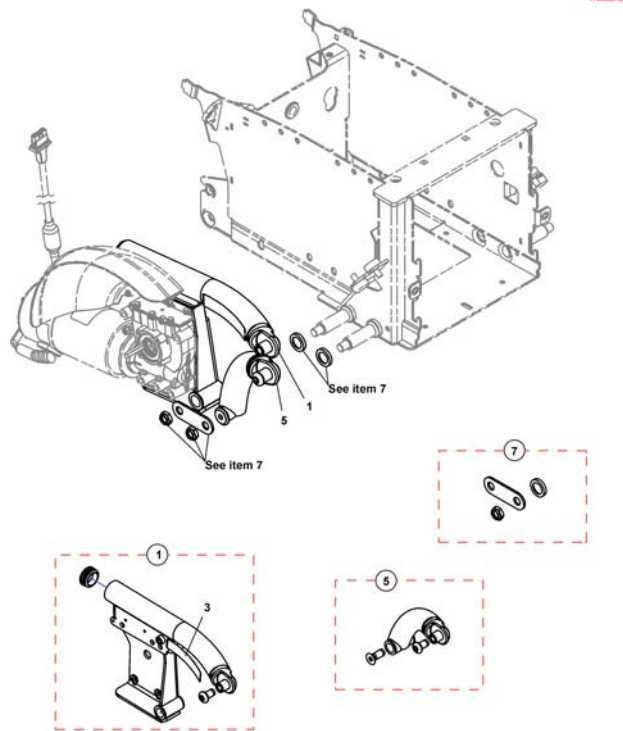

Pos.	Part number	Description	Valid from → until	Qty
1	SP1551713	Walking Beam Assembly (wet black) with TDX-SP Decal-Right		1
1	SP1557069	Walking Beam Assembly (electric red) with TDX-SP Decal-Right		1
1	SP1551714	Walking Beam Assembly (wet black) with TDX-SP Decal-Left		1
1	SP1557070	Walking Beam Assembly (electric red) with TDX-SP Decal-Left		1
3	SP1523167	Decal, TDX SP (Yellow)		1
5	SP1523198	Link, Lower Pivot (wet black)-Right		1
5	SP1563169	Link, Lower Pivot (electric red)-Right		1
5	SP1523199	Link, Lower Pivot (wet black)-Left		1
5	SP1563163	Link, Lower Pivot (electric red)-Left		1
7	SP1551715	Kit, Lower Pivot Link/4-Pole Motor Mounting Hardware		1


Rear Pivot Assembly

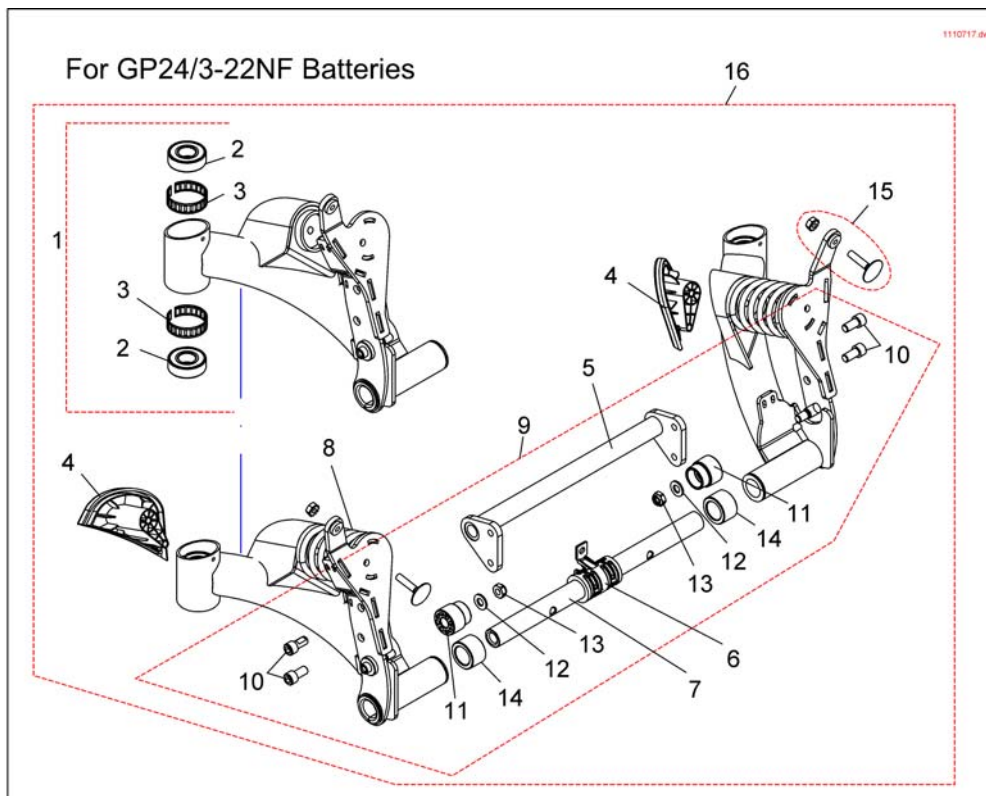

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523260	Kit, Swing Arm Assembly (electric red) with Bearings, Rear-Left		1
1	SP1523259	Kit, Swing Arm Assembly (electric red) with Bearings, Rear-Right		1
1	SP1564785	Kit, Swing Arm Assembly (wet black) with Bearings, Rear-Left		1
1	SP1564781	Kit, Swing Arm Assembly (wet black) with Bearings, Rear-Right		1
2	SP40117X006	Bearing (5/8 ")		1
3	SP1110717	Ring, Tolerance		1
4	SP1523089	Cap, Rear-Left		1
4	SP1523088	Cap, Rear-Right		1
5	SP1523083	Cross-Member, Pivot (TDX-SP Narrow Base)		1
5	SP1142287	No description available.		1
6	SP1523166	Bushing Assembly, Center		1
7	SP1523194	Bar, Pivot		1
7	SP1523197	Bar, Pivot		1
7	SP1155018	No description available.		1
8	SP1523002	Spring - yellow <i>replaced by SP1579643 (silver)</i>		1
8	SP1579643	Spring - silver <i>replaces SP1523002 (yellow)</i>		1
9	SP1523177	Kit, Pivot Assembly Hardware <i>included parts 10-14</i>		1
13	SP1522937	Locknut (5/16-18)		1
15	SP1523144	Kit, Elevator Bolt w/Keps Nut (1/4-20)		1
16	SP1523261	Rear Pivot Assembly (electric red) (ForGP24/3-22NF Battery Box) (wide base)		1
16	SP1564779	Rear Pivot Assembly (wet black) (ForGP24/3-22NF Battery Box) (wide base)		1
16	SP1551708	Rear Pivot Assembly (electric red) (For22NF Battery Box) (Narrow base)		1
16	SP1564780	Rear Pivot Assembly (wet black) (For 22NF Battery Box) (Narrow base)		1

Rear Pivot Assembly (Mounting)


1149285 Rev

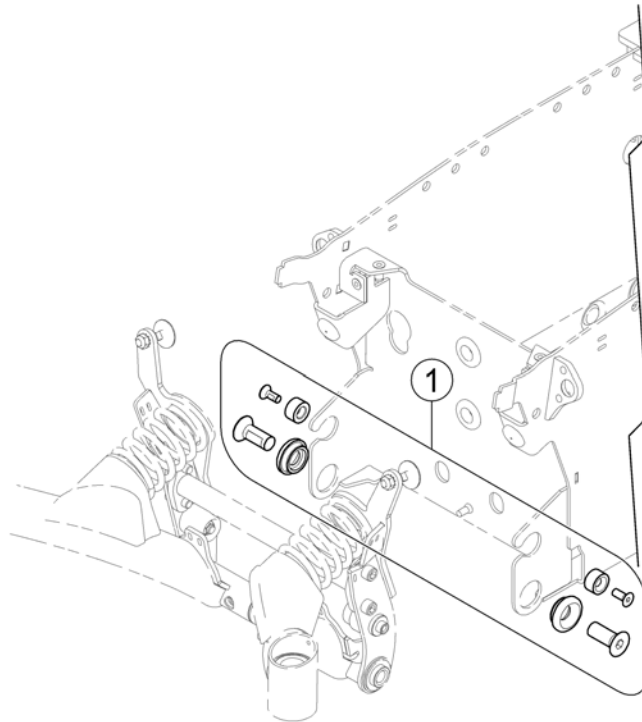

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523193	Kit, Rear Pivot Assembly Mounting Hardware		1



Cylinders, Locking Gas

1144661.00

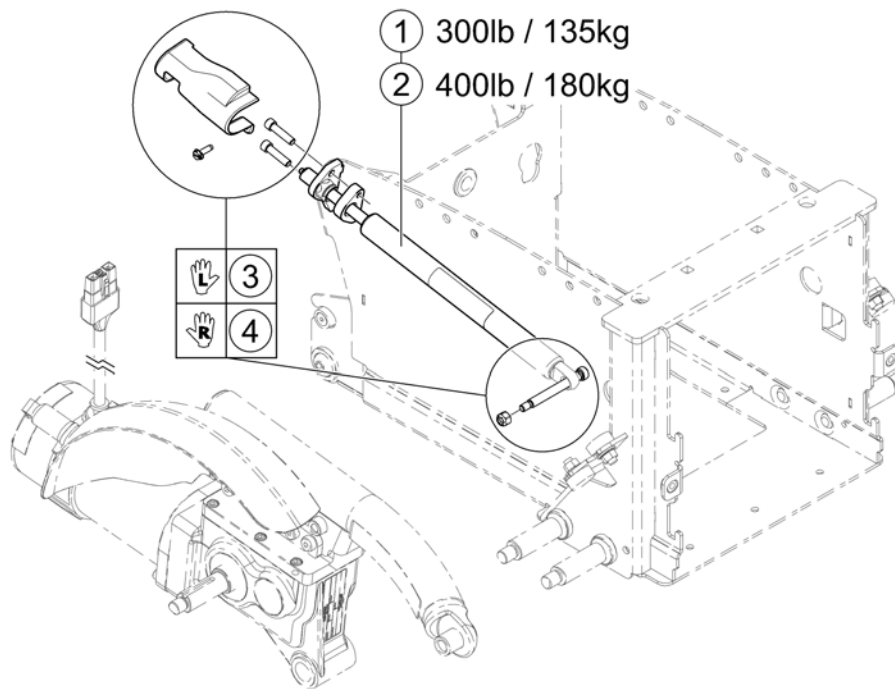


Pos.	Part number	Description	Valid from → until	Qty
1	SP1523162	Cylinder, Locking Gas (300 Lb. User Weight Limit)		1
2	SP1523158	Cylinder, Locking Gas (400 Lb. User Weight Limit)		1
3	SP1523149	Kit, Locking Gas Cylinder Mounting Hardware -Left		1
4	SP1523148	Kit, Locking Gas Cylinder Mounting Hardware -Right		1

Fender

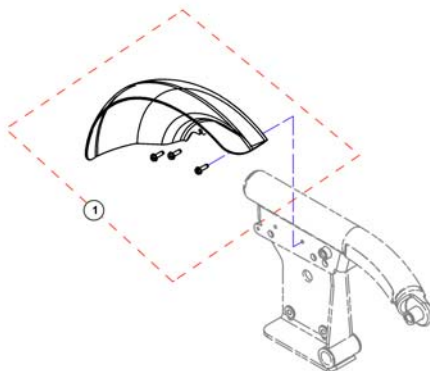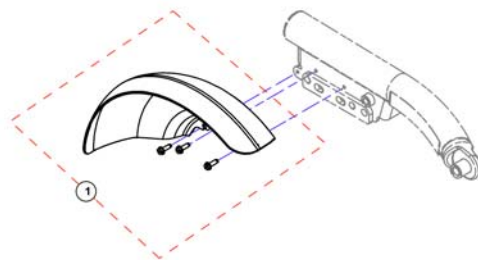

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523170	Kit, Fender Assembly - Right		1
1	SP1523171	Kit, Fender Assembly - Left		1

Shrouds and Hardware

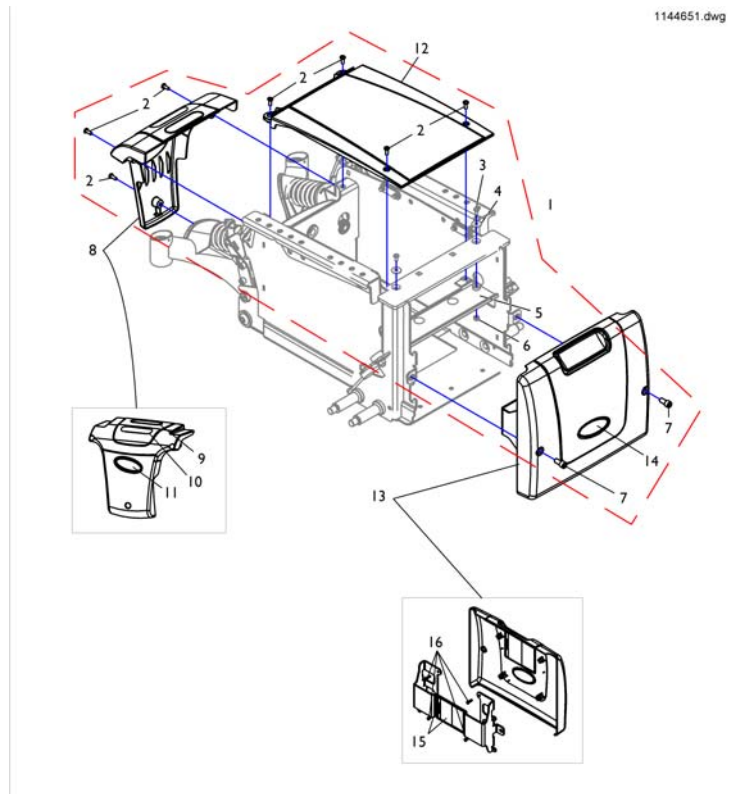


1144651.dwg

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523187	Kit, Shroud Hardware (ForGP24/3-22NF Batteries)		1
1	SP1144651	Kit, Shroud Hardware for 22NF Batteries (narrow base)		1
6	SP1522936	Locknut (10-32)		1
7	SP1536212	knurled screw 5/16"		1
8	SP1523183	Shroud, Rear w/Decals (GP24/3-22NF Batteries) C Used For TRRO/TRBKTS Option		1
8	SP1523155	Shroud, Rear w/Decal (TRRO/TRBKTS) (narrow base)		1
12	SP1523182	Shroud, Center (GP24 Batteries) (For TRRO/TRBKTS Option Also)		1
12	SP1523153	Shroud, Center (TRRO/TRBKTS) (narrow base)		1
13	SP1523189	Front Shroud and Bracket Assembly (ForGP24 Batteries)		1
13	SP1144642	Front Shroud and Bracket Assembly, for 2-22NF Batteries (narrow base)		1
15	SP1523190	Bracket w/Decal, Front Shroud (ForGP24 Batteries)		1
16	SP1523003	Screw, Phillips Head (8-16 x5/8")		1

Battery Box Hardware


1140071.00

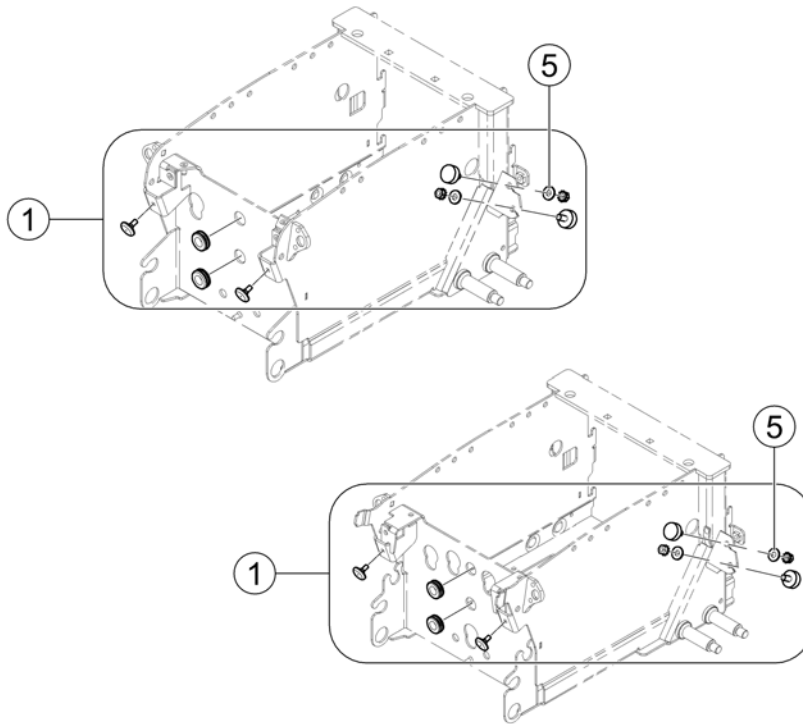

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523179	Kit, Battery Box, Rubber Bumper & Grommet Hardware		1
5	SP1522968	Washer (5/16 x3/4 x1/16")		1

Seat

SPCTDX_SP20.00

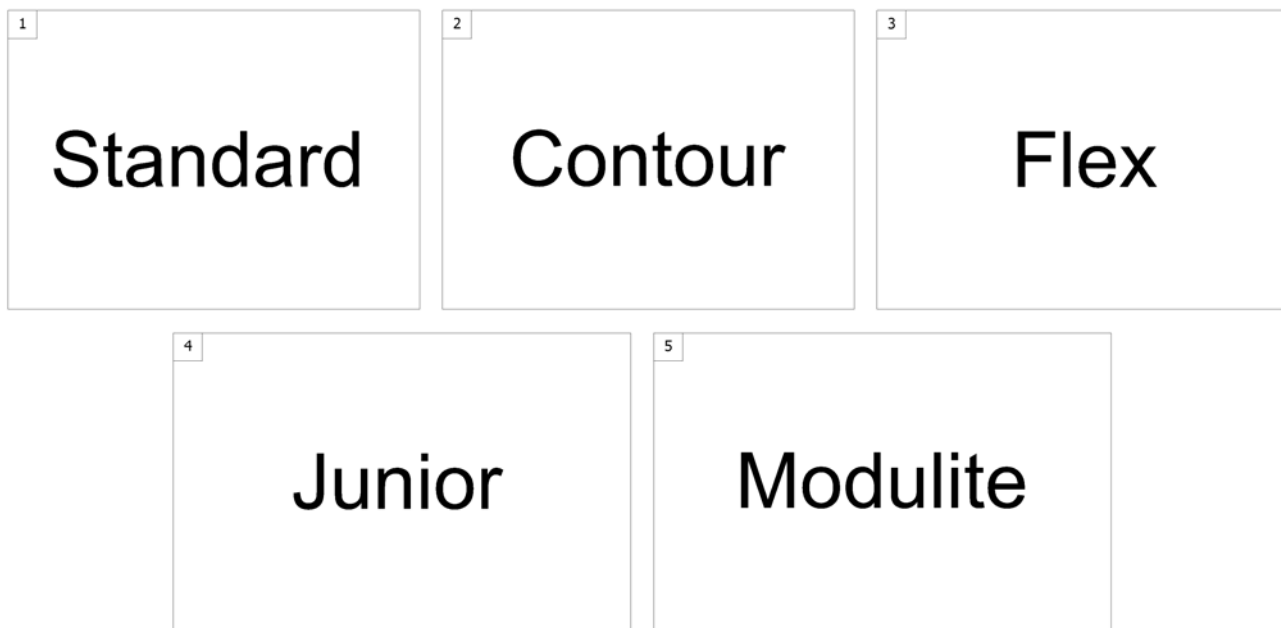


Pos.	Part number	Description	Valid from → until	Qty
1	-	Standard Seat		1
2	-	Contour Seat		1
3	-	Flex 2 Seat		1
4	-	Junior Seat		1
5	-	Modulite Seat		1

Standard Seat


Pos.	Part number	Description	Valid from → until	Qty
-		Powered Tilt 50° & manuell		1
-		Tiltmodule 28°		1
-		Lifter (incl. 28°-Tilt)		1
-		Bellow		1
-		Seat frames for Balle modules		1
-		Adapter sheets		1
-		Seatplates		1
-		Seat Cushion for Standard Seat		1

Powered Tilt 50° & manuell

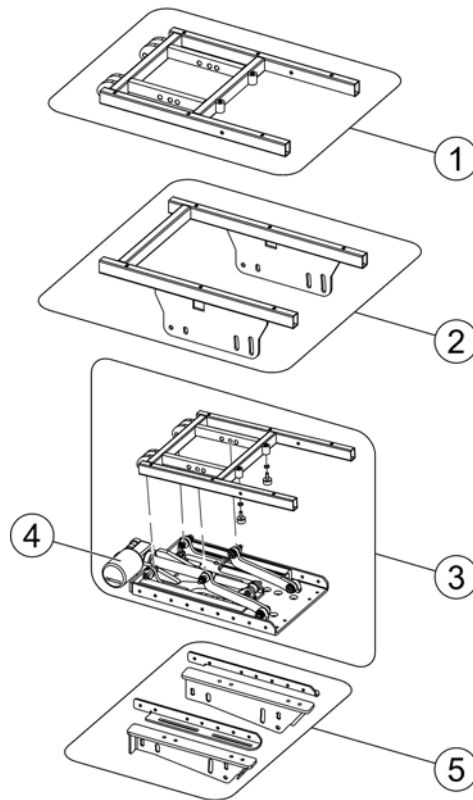


Pos.	Part number	Description	Valid from → until	Qty
-	SP1535829	Mercury Switch for tilt up to 50°		1
1	SP1513344	Seatframe for Electric Tilt 50°		1
2	SP1513657	Seatframe for Manual Tilt (292mm)		1
2	SP1548900	Seatframe for Manual Tilt (254mm)		1
3	SP1528314	CG-Tilt TDX-SP with Seatframe		1
4	SP1143801	Tilt-Actuator		1
5	SP1580051	Angle Bracket for CG-Tilt (pair) <i>Incl. screws / avec vis / inkl. Schrauben</i>		1

Tiltmodule 28°

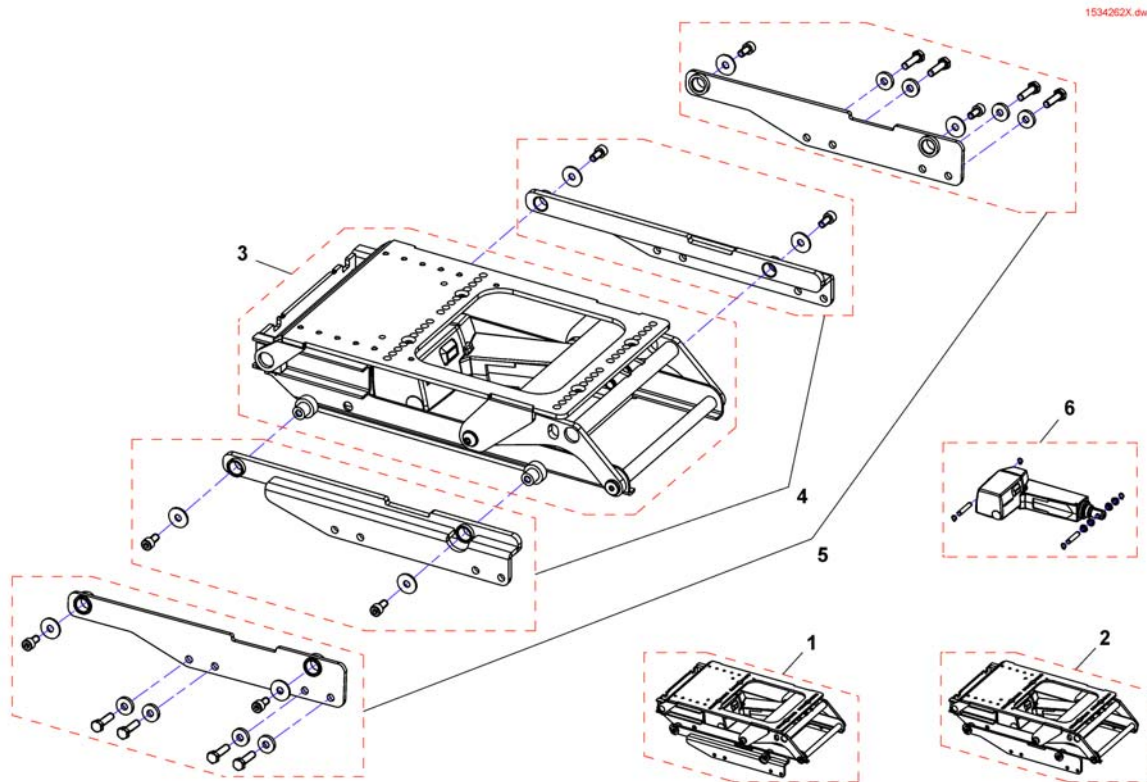


Pos.	Part number	Description	Valid from → until	Qty
1	SP1534262	Seattilt 0-28°, cpl.		1
2	SP1548546	Seattilt 0-28°, cpl., Narrowbase		1
3	SP1533063	Tilt unit 0-28°		1
4	SP1536176	Guiding rails, cpl.		1
5	SP1551035	Guiding rails, cpl., Narrowbase		1
6	SP1544888	Actuator		1

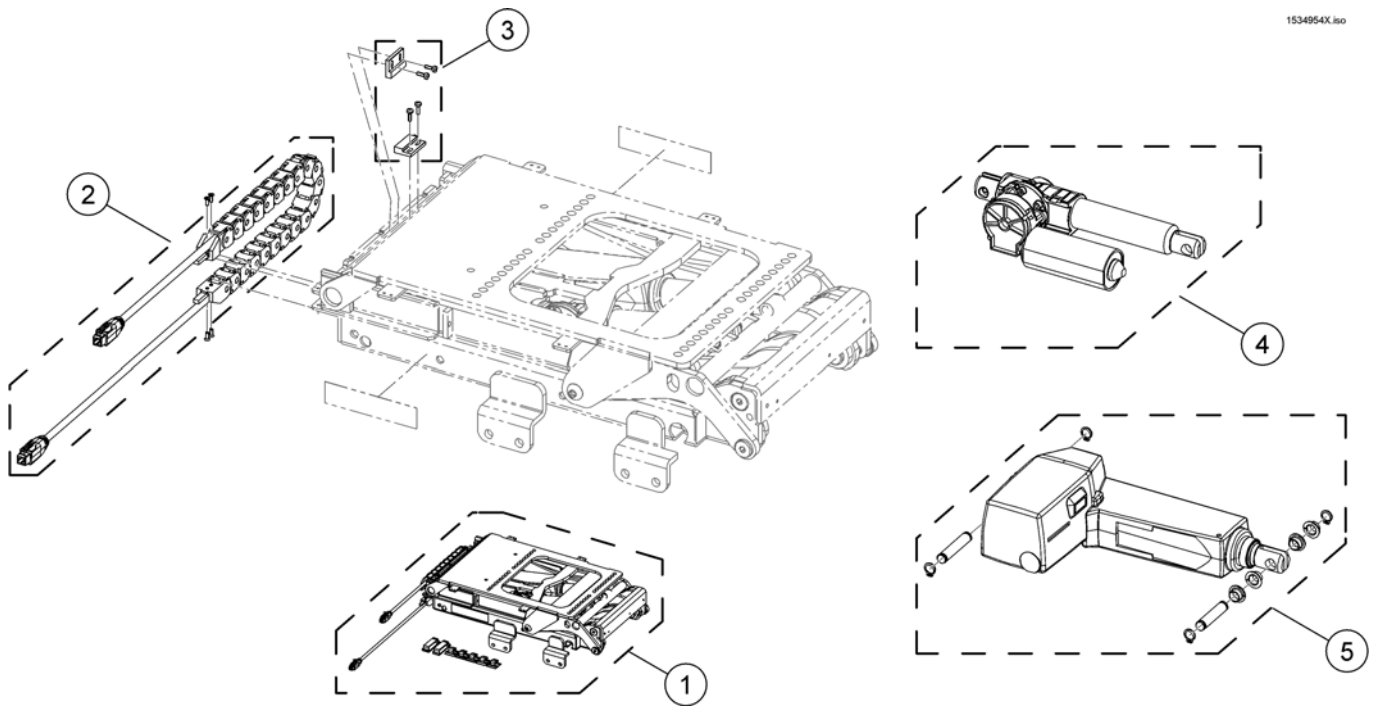


Pos.	Part number	Description	Valid from → until	Qty
-	SP1533562	Actuator cable ACS2 (~1570mm)		1
-	SP1571139	Tilt actuator cable ACS2 (~920mm)		1
1	SP1534954	Lifter. incl. 28°-Tilt, cpl.		1
2	SP1535848	Cable carrier with cable		1
3	SP1536184	Speed reduction sensor		1
4	SP1533557	Tilt actuator, ACS2		1
5	SP1533616	Lifter actuator		1

Bellow

1545673X.dwg

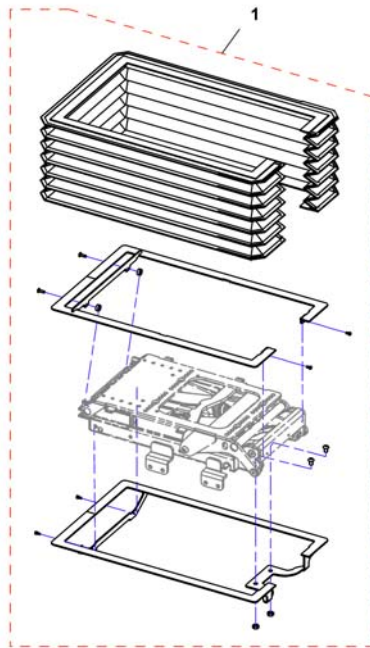




Pos.	Part number	Description	Valid from → until	Qty
	SP1545673	Bellow		1

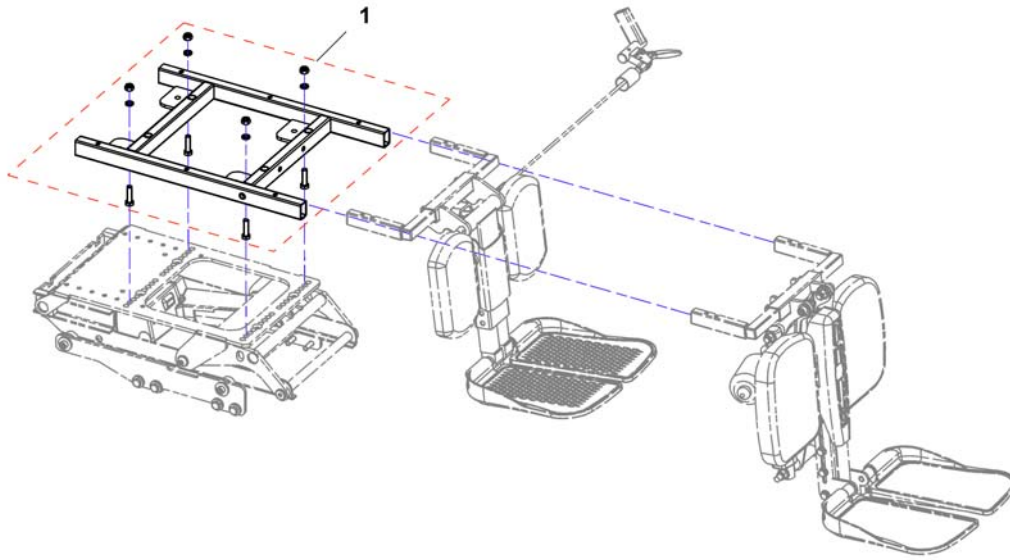

Pos.	Part number	Description	Valid from → until	Qty
1	SP1552400	Seatframes for Balle modules		1

Adapter sheets



1534951X.dwg

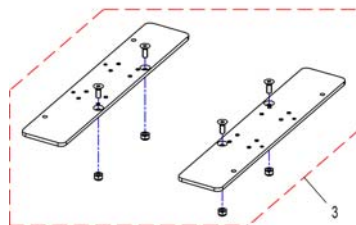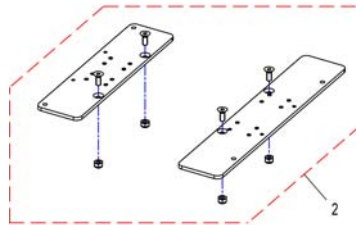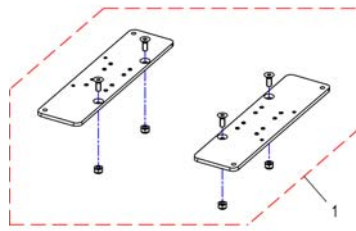

Pos.	Part number	Description	Valid from → until	Qty
1	SP1534951	Adaptersheet for tilt & lifter SW39		1
2	SP1534952	Adaptersheet for tilt & lifter SW44		1
3	SP1534953	Adaptersheet for tilt & lifter SW49		1

Seatplates

1420459X.dwg

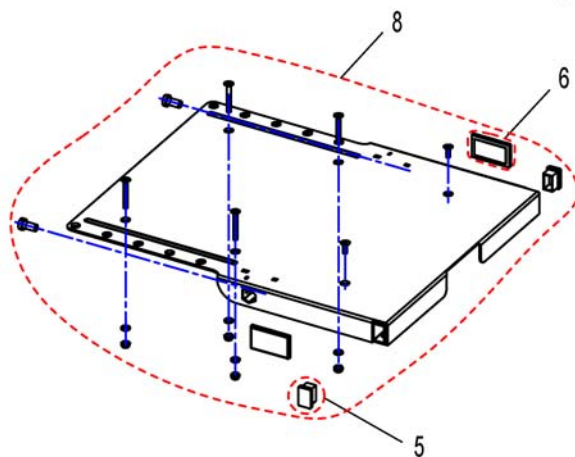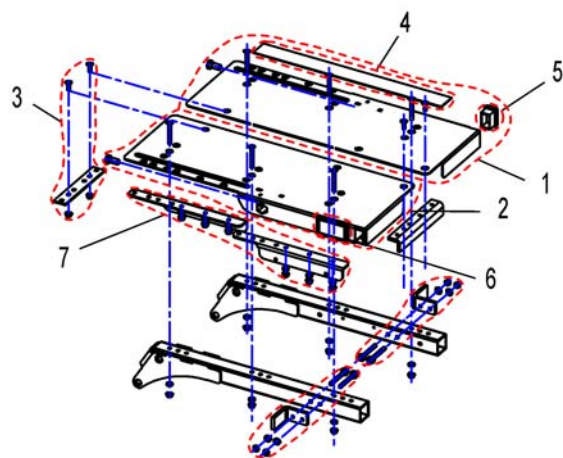

Kontour




Flex



Standard



Pos.	Part number	Description	Valid from → until	Qty
1	SP1420458	Seat Plate LH		1
2	SP1420457	Seat Plate RH		1
3	SP1420496	Joining Plate		1
4	SPU21087	Velcro		1
5	SPL01060	Darning		1
6	SPP31298	Reflector, yellow		1
7	1510637	Stiffing Kit		1
8	SP1440623	Seat plate, cpl.		1

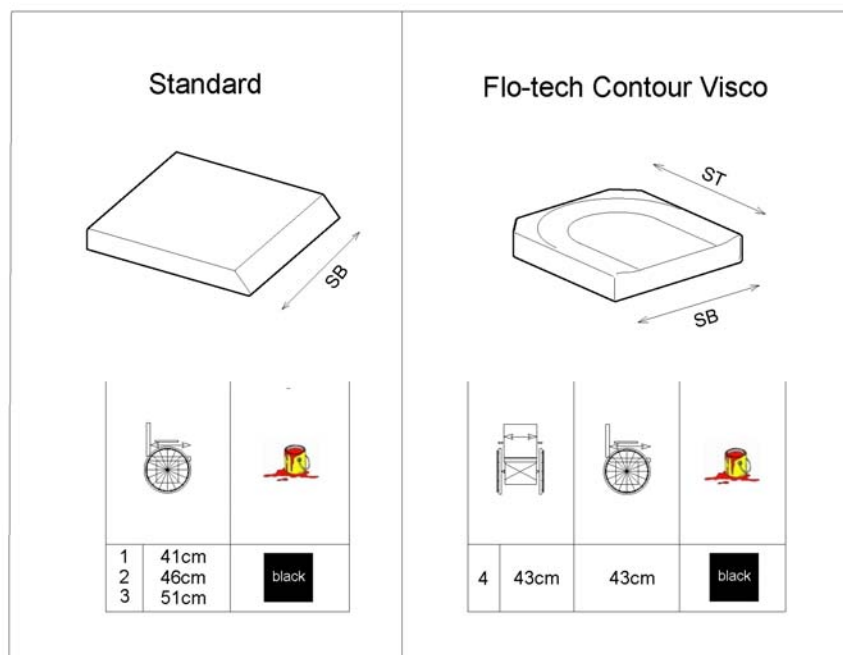

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard SD 410mm [1336+0326]		1
2	1420461	Seat Cushion Standard SD 460mm [1336+0327]		1
3	1420462	Seat Cushion Standard SD 510mm [1336+0330]		1
4	1501113	Seat Cushion Flo-tech Contour Visco SW 43cm x SD 43cm [1901]		1

Contour Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Powered Tilt 50° & manuell		1
-		Tiltmodule 28°		1
-		Lifter (incl. 28°-Tilt)		1
-		Bellow		1
-		Seat frames for Balle modules		1
-		Adapter sheets		1
-		Seatplates		1
-		Seat Cushion for Contour Seat		1


Seat Cushion for Contour Seat

1573660.iso

Contour



Flo-tech Contour Visco

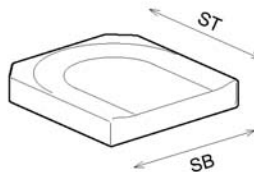

1	41cm		black
2	46cm		

3	43cm	43cm	black
4	45,5cm	53cm	
5	48cm	40,5cm	
6	48cm	43cm	


Pos.	Part number	Description	Valid from → until	Qty
1	1422072	Cushion Contour SW43cm, black [1337]		1
2	1422074	Cushion Contour SW48cm, black [1335]		1
3	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm [1901]		1
4	SP1500993	Seat Cushion Flo-tech Contur Visco 45.5x53cm [1902]		1
5	SP1501116	Seat Cushion Flo-tech Contur Visco 48x40.5cm [1903]		1
6	SP1501117	Seat Cushion Flo-tech Contur Visco 48x43cm [1904]		1

Flex 2 Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Powered Tilt 50° & manuell		1
-		Tiltmodule 28°		1
-		Lifter (incl. 28°-Tilt)		1
-		Bellow		1
-		Seat frames for Balle modules		1
-		Adapter sheets		1
-		Seatplates		1
-		Seat Cushion for Flex Seat		1

Seat Cushion for Flex Seat

1573688 r00

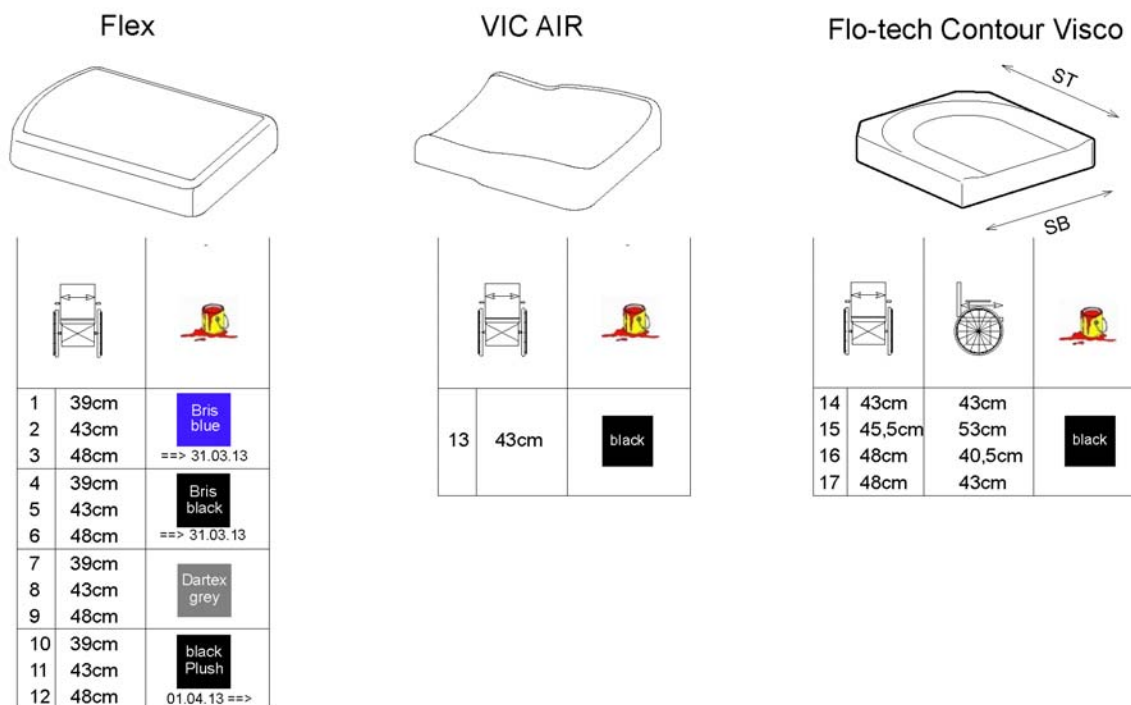


Pos.	Part number	Description	Valid from → until	Qty
1	1422529	Cushion Flex SW39cm, blue	→ 2013-03-31	1
2	1422530	Cushion Flex SW43cm, blue	→ 2013-03-31	1
3	1422532	Cushion Flex SW48cm, blue	→ 2013-03-31	1
4	1436561	Cushion Flex SW39cm, black	→ 2013-03-31	1
5	-	No longer available - please choose Black Plush <i>replaced by SP1436562-TR35</i>	→ 2013-03-31	0
6	-	No longer available - please choose Black Plush <i>replaced by SP1436563-TR35</i>	→ 2013-03-31	0
7	5252582-TR23	SEAT CUSHION SB 39; DARTEX		1
8	5252583-TR23	CUSHION TROMB 410X530, SEAT, TR23		1
9	5252584-TR23	CUSHION TROMB 460X530, SEAT, TR23		1
10	SP1436561-TR35	Seat cushion SW39cm, black Plush	2013-04-01 →	1
11	SP1436562-TR35	Seat cushion SW43cm, black Plush	2013-04-01 →	1
12	SP1436563-TR35	Seat cushion SW48cm, black Plush	2013-04-01 →	1
13	1422531	Cushion Flex SW43cm, VIC AIR		1
14	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm		1
15	SP1500993	Seat Cushion Flo-tech Contur Visco 45.5x53cm		1
16	SP1501116	Seat Cushion Flo-tech Contur Visco 48x40.5cm		1
17	SP1501117	Seat Cushion Flo-tech Contur Visco 48x43cm		1

Junior Seat


Pos.	Part number	Description	Valid from → until	Qty
-		Powered Tilt 50° & manuell		1
-		Tiltmodule 28°		1
-		Lifter (incl. 28°-Tilt)		1
-		Bellow		1
-		Seat frames for Balle modules		1
-		Adapter sheets		1
-		Seatplates		1
-		Seat Cushion for Junior Seat		1

Seat Cushion for Junior Seat

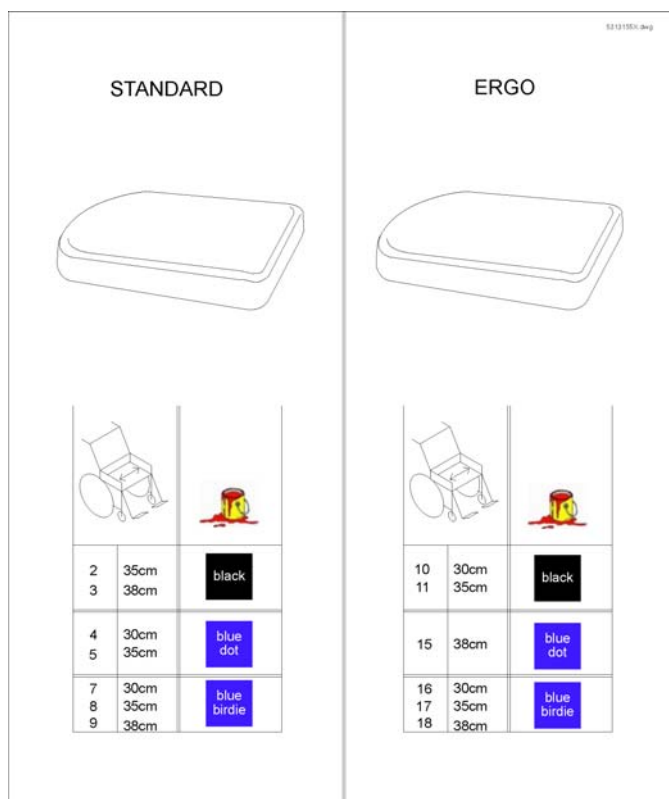

Modulite Seat


1574046.iss

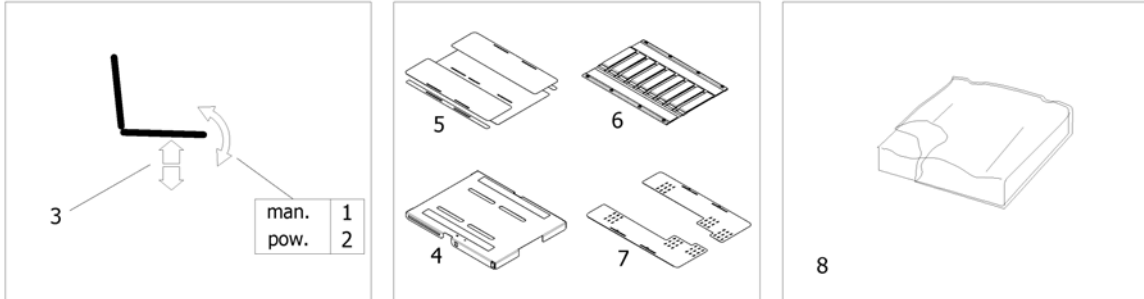


Pos.	Part number	Description	Valid from → until	Qty
1	-	man. settable Tilt [1091]		1
2	-	powered Tilt 0°-30° [1093]		1
3	-	Lifter [1094]		1
4	-	One Piece Seat Plate [2110]		1
5	-	Telescopic Seat Frame and Plate [2111]		1
6	-	Telescopic Seat Frame and Sling Seat [2112]		1
7	-	Telescopic Seat Frame and Universal Adapter [2109]		1
8	-	Seat Cushion		1

man. settable Tilt

1577301 iso

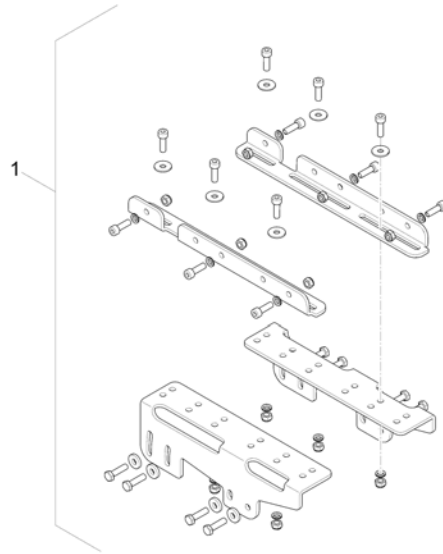


Pos.	Part number	Description	Valid from → until	Qty
1	SP1563229	Mounting Rails		1

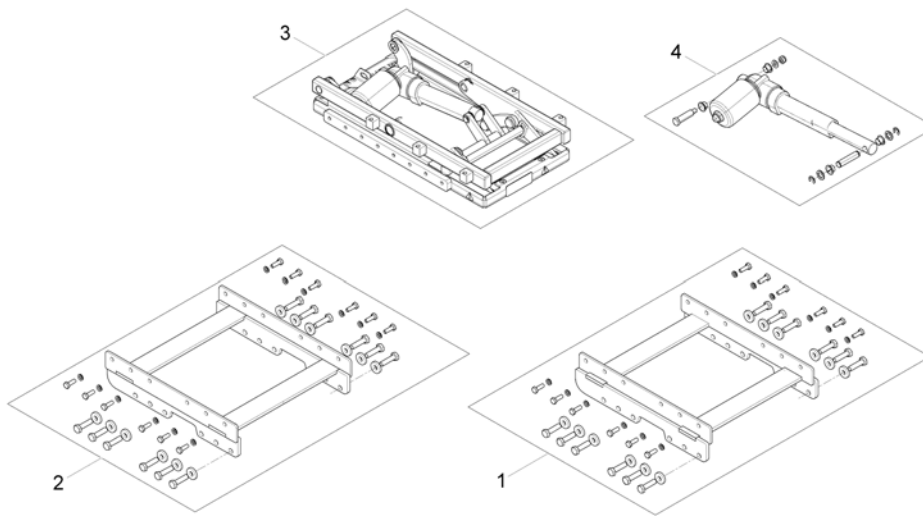

Pos.	Part number	Description	Valid from → until	Qty
1	SP1568426	Lifter- /Tilt Bracket for wide Base		1
2	SP1568424	Lifter- /Tilt Bracket for narrow Base		1
3	SP1558714	Tilt unit		1
4	SP1571809	Actuator		1
-	SP1570291	Adapter Cable		1

Lifter


1577303 iso

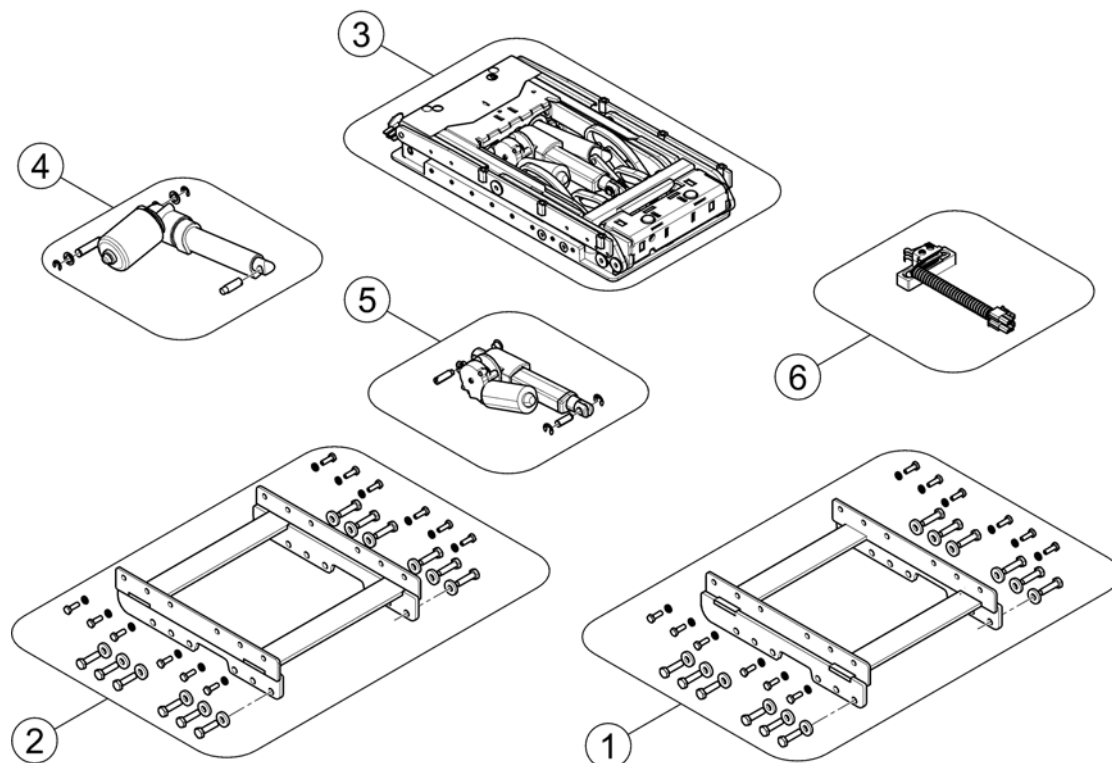

Pos.	Part number	Description	Valid from → until	Qty
-	SP1570291	Adapter Cable <i>(2x for wide Base / 1x for Narrow Base)</i>		1
-	SP1576062	Extension Cable <i>(only for narrow Base)</i>		1
-	SP1570294	Extension Cable		1
-	SP1533561	Light Cable w. DCI		1
1	SP1568426	Lifter- /Tilt Bracket for wide Base		1
2	SP1568424	Lifter- /Tilt Bracket for narrow Base		1
3	SP1558715	Lifter- / Tilt unit		1
4	SP1571823	Lifter actuator		1
5	SP1571822	Tilt actuator		1
6	SP1577762	Slow-Stop Switchcable with Mounting		1

One Piece Seat Plate


1571064.000

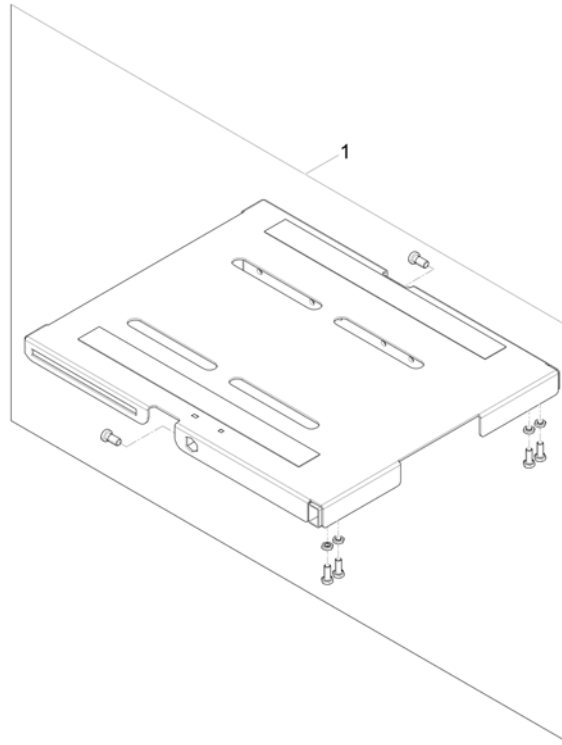

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571297	One Piece Seat Plate		1

Telescopic Seat Frame and Plate


1572312 iso

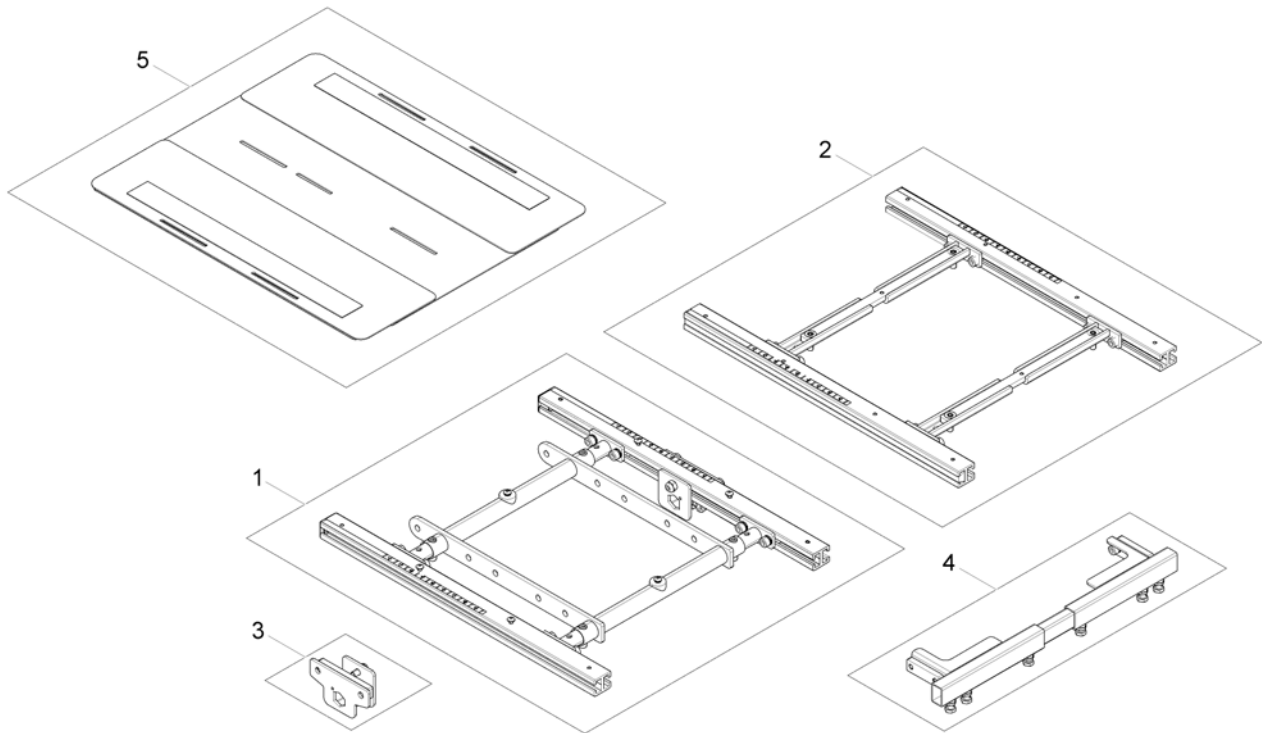

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 12° / 20°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Seat Plate		1

Telescopic Seat Frame for Tilt 12° / 20°


1572008.iss

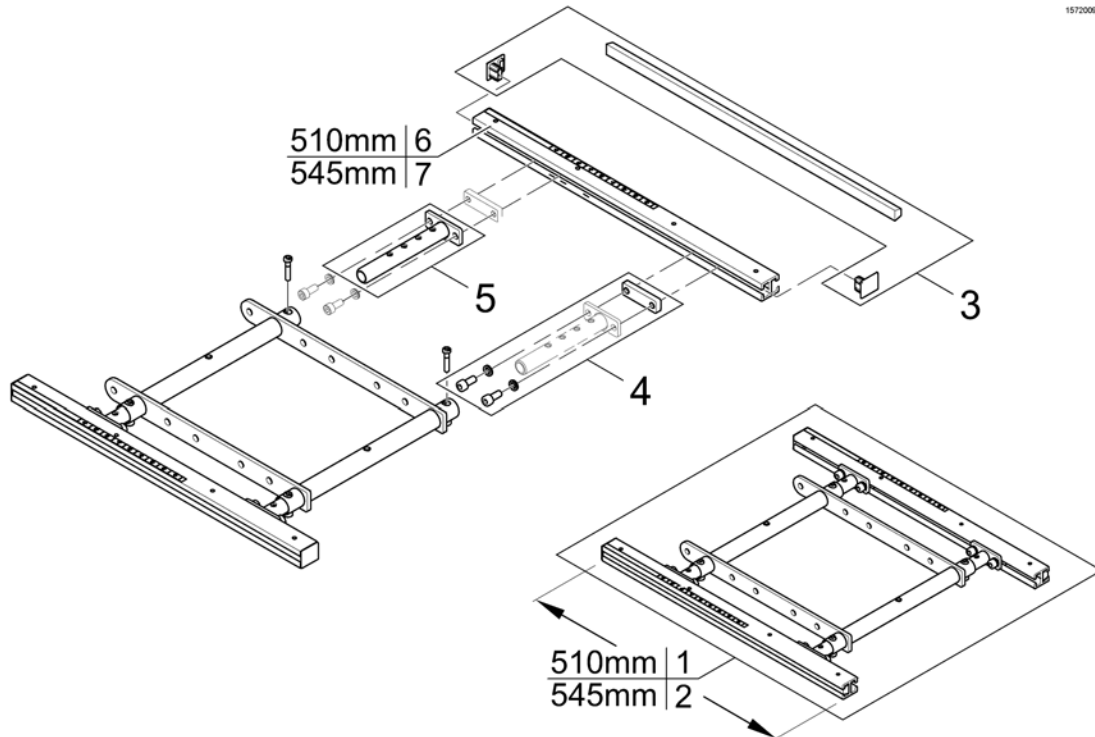

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572977	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582216	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1554646	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1

Telescopic Seat Frame for Tilt 30° / Lifter


1572004 Rev

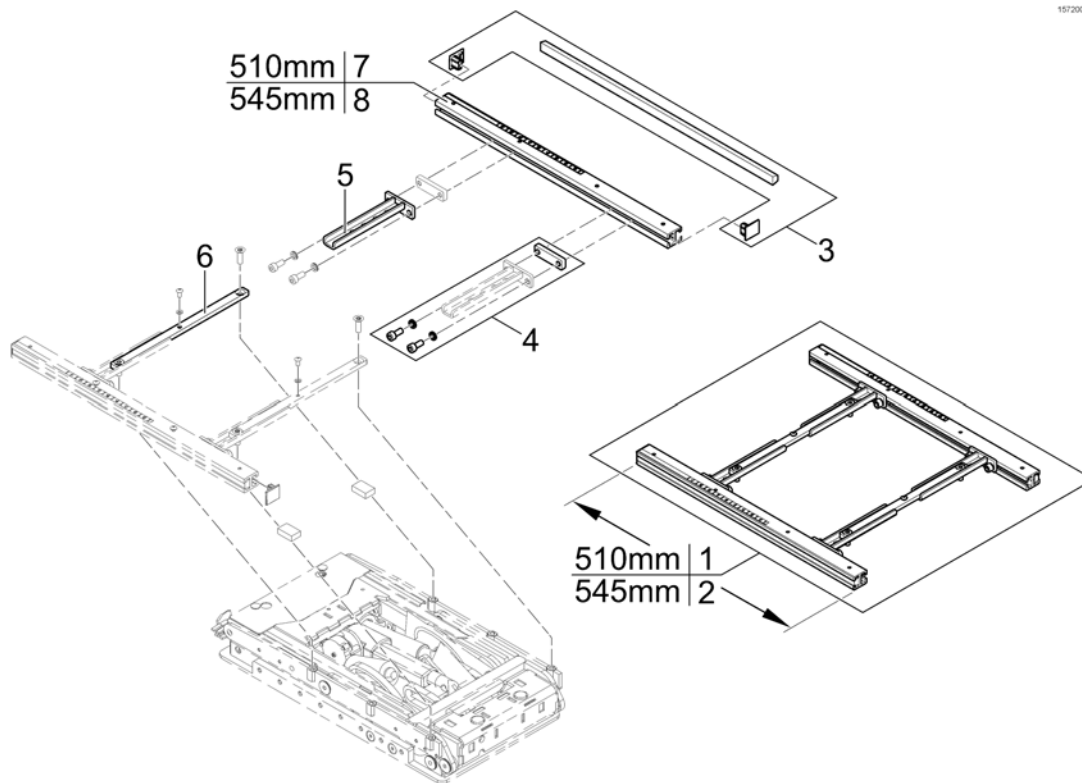

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572992	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582219	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1567007	Seat Frame Adapter		1
6	SP1567008	Middle Rail		2
7	SP1588443	Seat Rails 510mm (2pcs kit)		1
8	SP1588444	Seat Rails 545mm (2pcs kit)		1

Armrest bracket


1571834 Rev

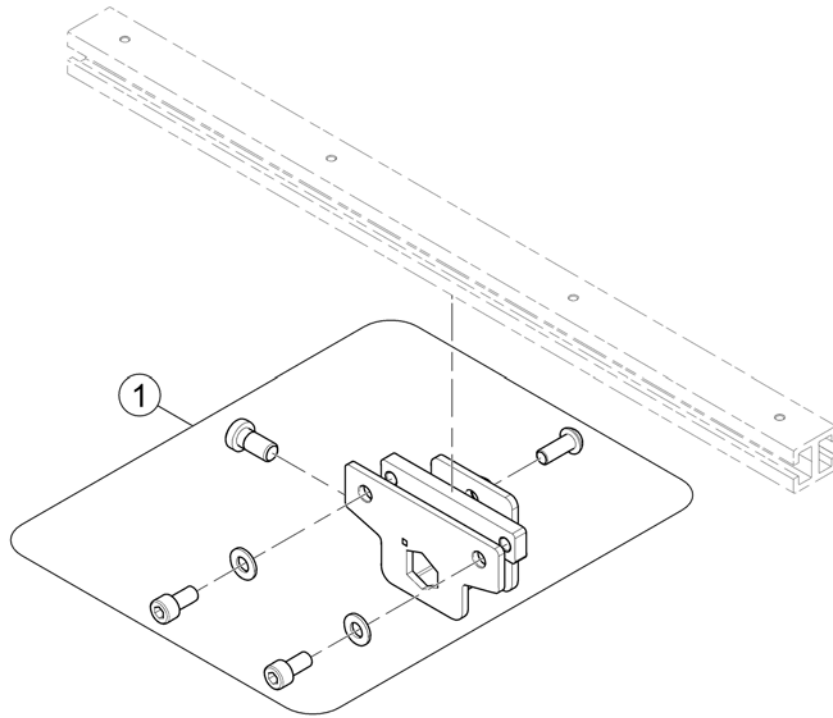

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570189	Armrest bracket		1

Legrest Support


1572021.000

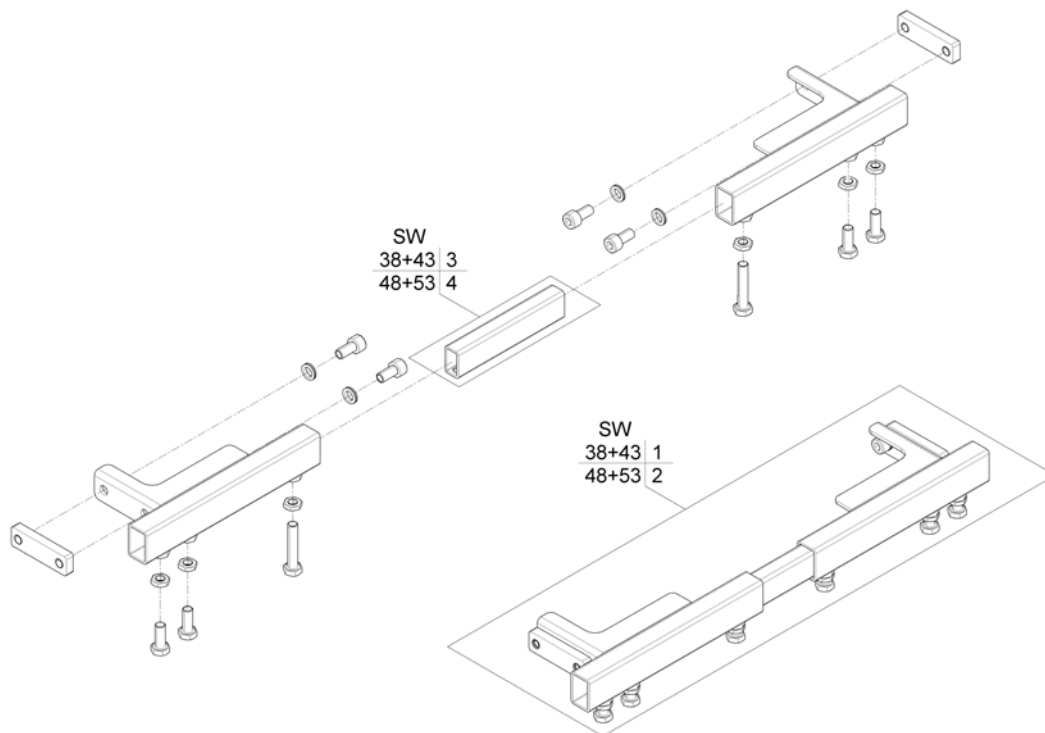

Pos.	Part number	Description	Valid from → until	Qty
1	SP1558935	Legrest Support SW38+43, cpl.		1
2	SP1559294	Legrest Support SW48+53, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

Seat Plate


1572310.00

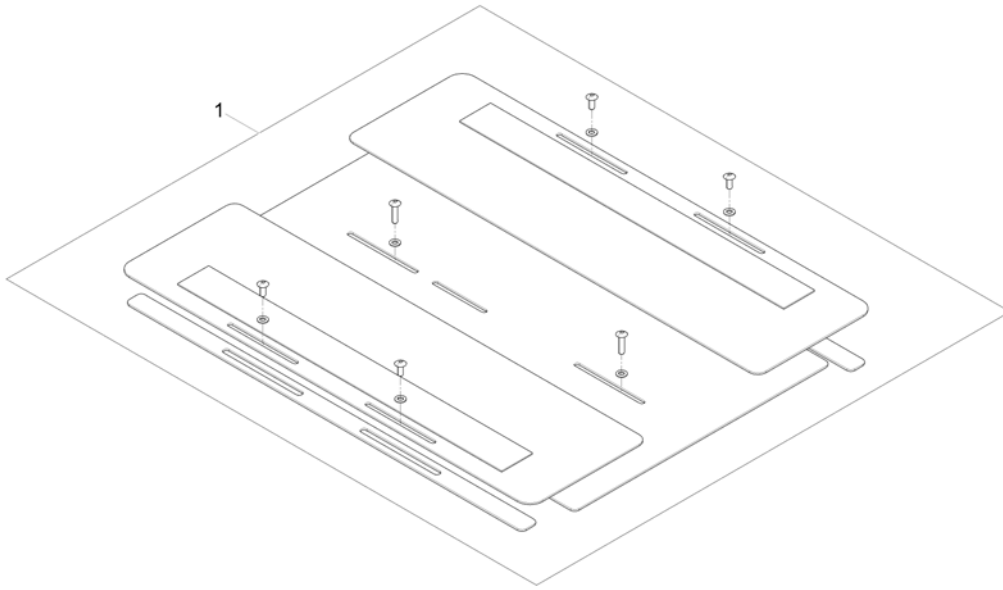

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572199	Seat Plate 510mm, cpl.		1

Telescopic Seat Frame and Sling Seat


1572311.00

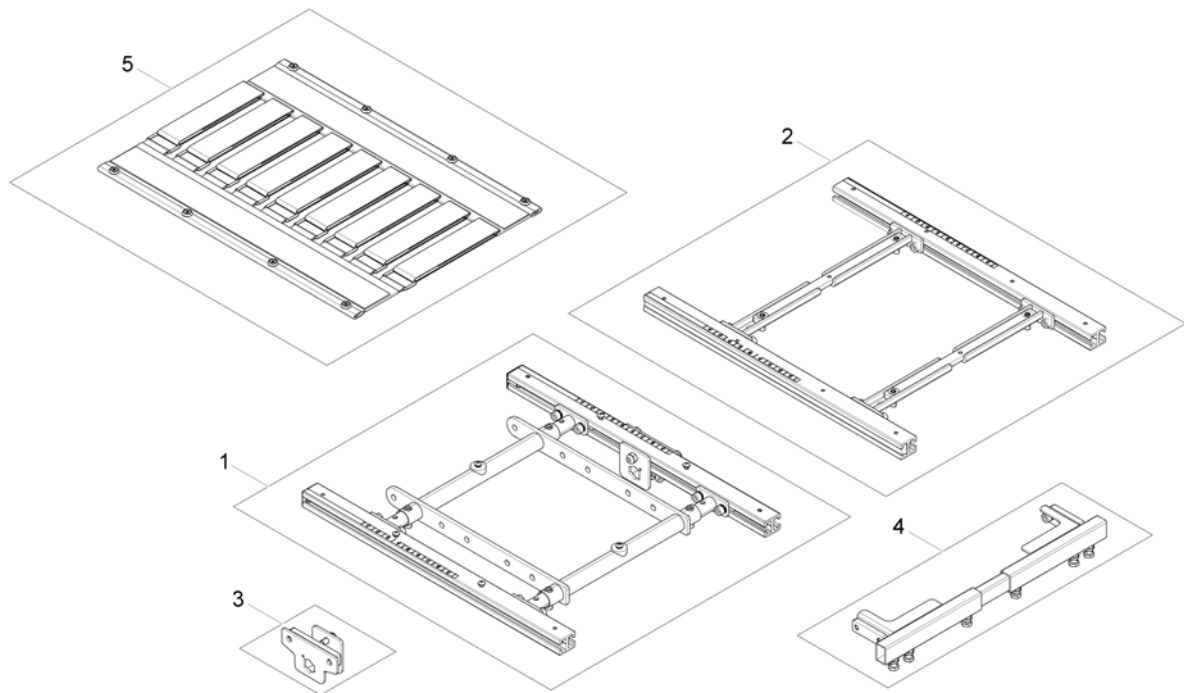

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 12° / 20°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Sling Seat		1

Telescopic Seat Frame for Tilt 12° / 20°


1571950.iss

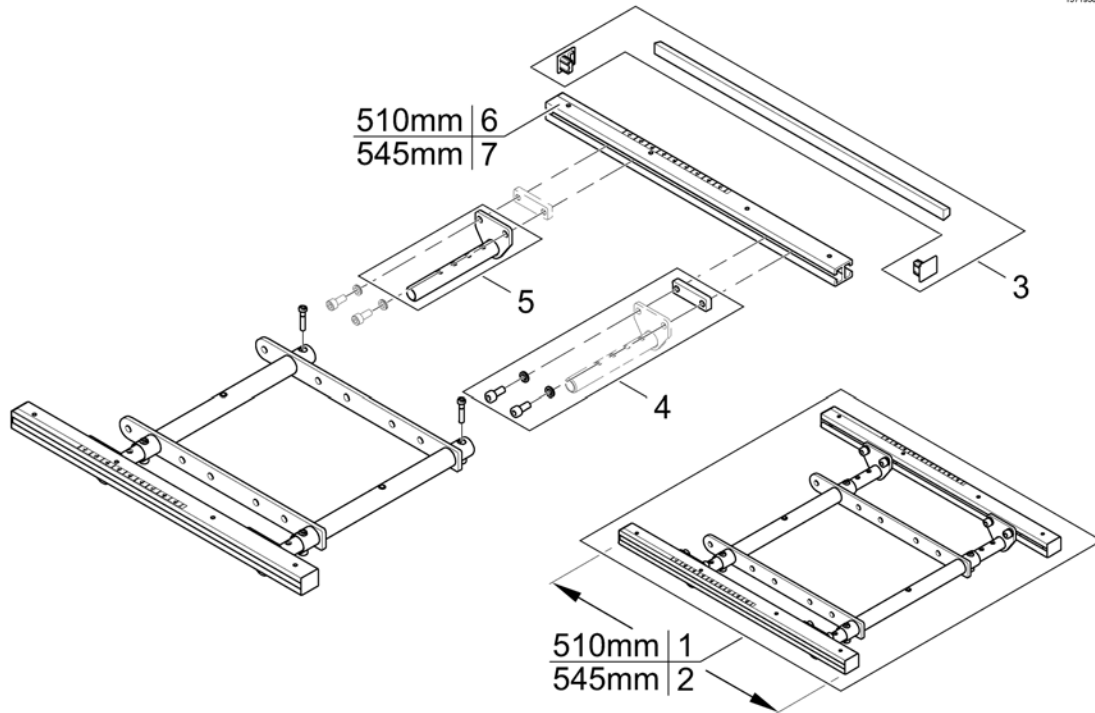

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572978	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582217	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1555400	Adjustment Tube		1
6	SP1588443	Seat Rails 510mm (2pcs kit)		1
7	SP1588444	Seat Rails 545mm (2pcs kit)		1

Telescopic Seat Frame for Tilt 30° / Lifter


1571832 Rev

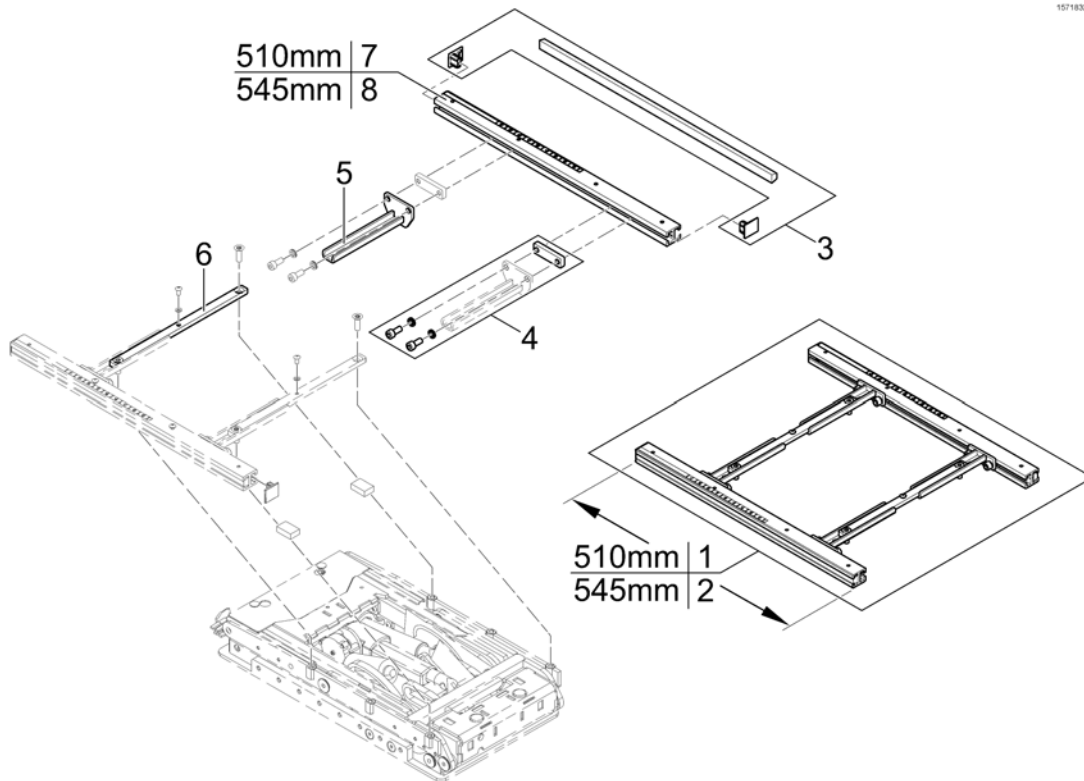

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572993	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582230	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1571810	Sealing strap with plugs		1
4	SP1571811	Groove stone with mounting parts		1
5	SP1567720	Seatframe Adapter		1
6	SP1567008	Middle Rail		1
7	SP1588443	Seat Rails 510mm (2pcs kit)		1
8	SP1588444	Seat Rails 545mm (2pcs kit)		1

Legrest Support


1572030.000

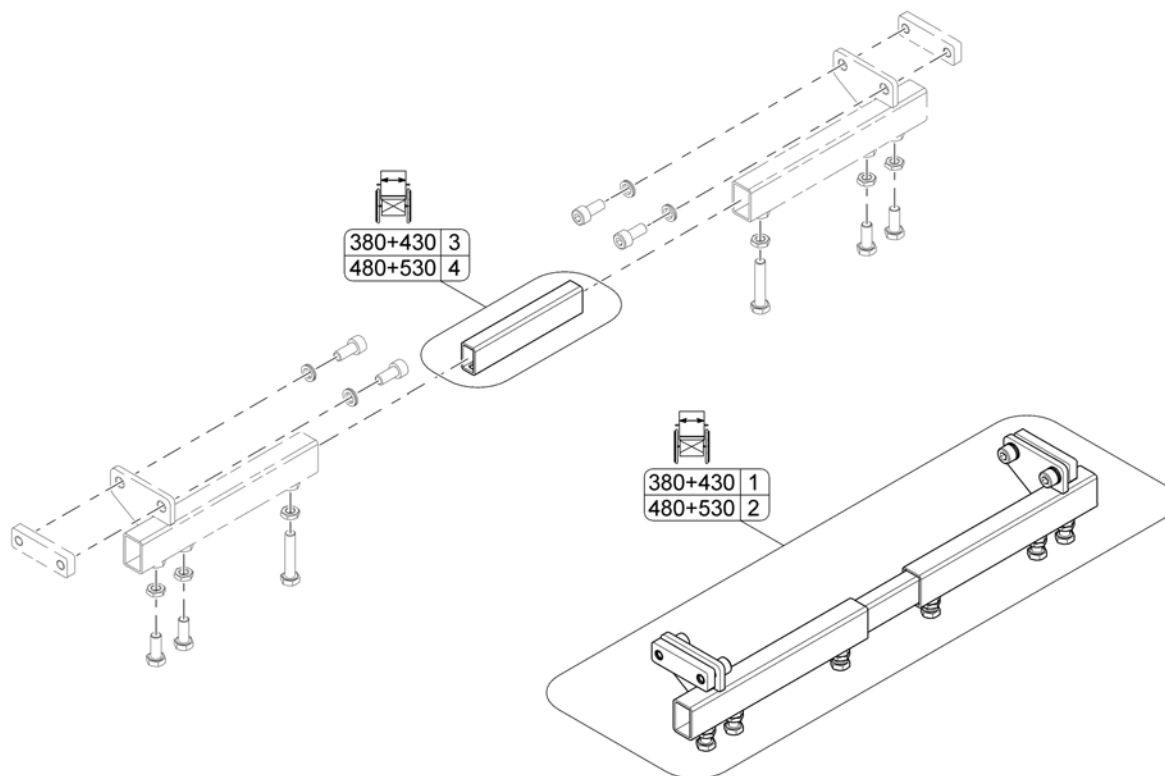

Pos.	Part number	Description	Valid from → until	Qty
1	SP1559305	Legrest Support SW380+430mm, cpl.		1
2	SP1559307	Legrest Support SW480+530mm, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

Sling Seat


151176290

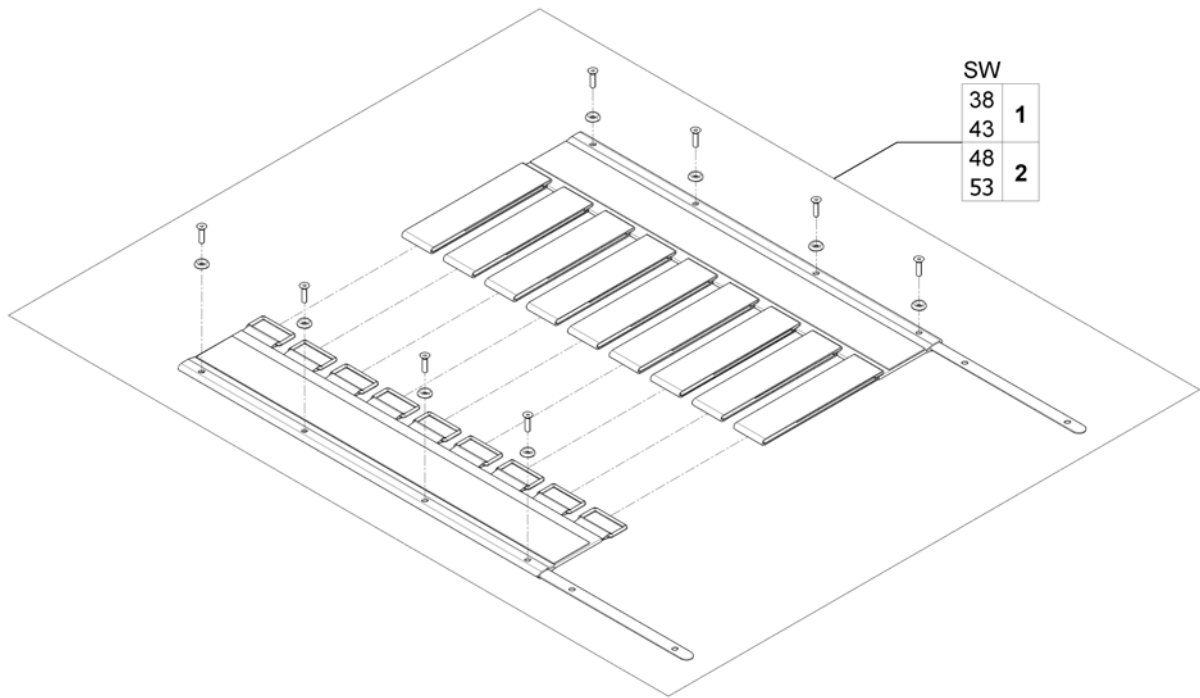

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557877	Sling Seat SW380/430, 510mm		1
2	SP1555911	Sling Seat SW480/530, 510mm		1

Telescopic Seat Frame and Universal Adapter


1581864 iso

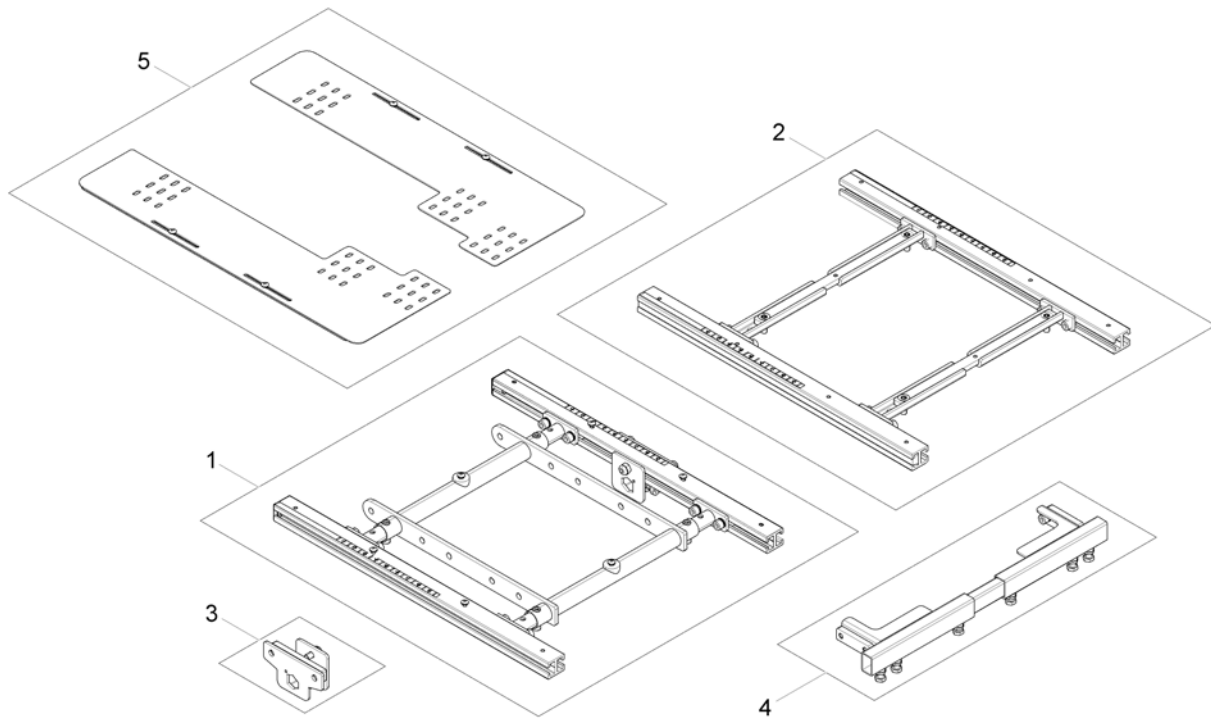

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for Tilt 12° / 20°		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest bracket		1
4	-	Legrest Support		1
5	-	Adapter Kit for Universal Seat		1

Adapter Kit for Universal Seat


1581863.iso

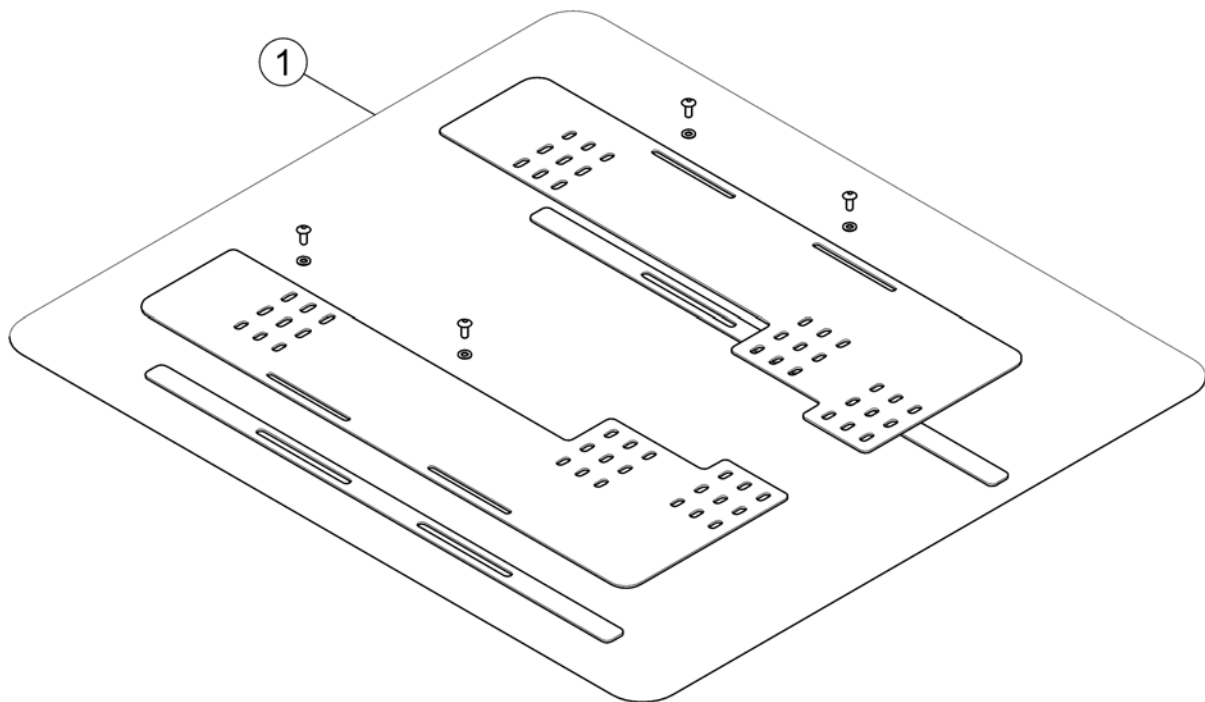

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580750	Adapter Kit for Universal Seat 510mm		1

Seat Cushion

1573000 iso

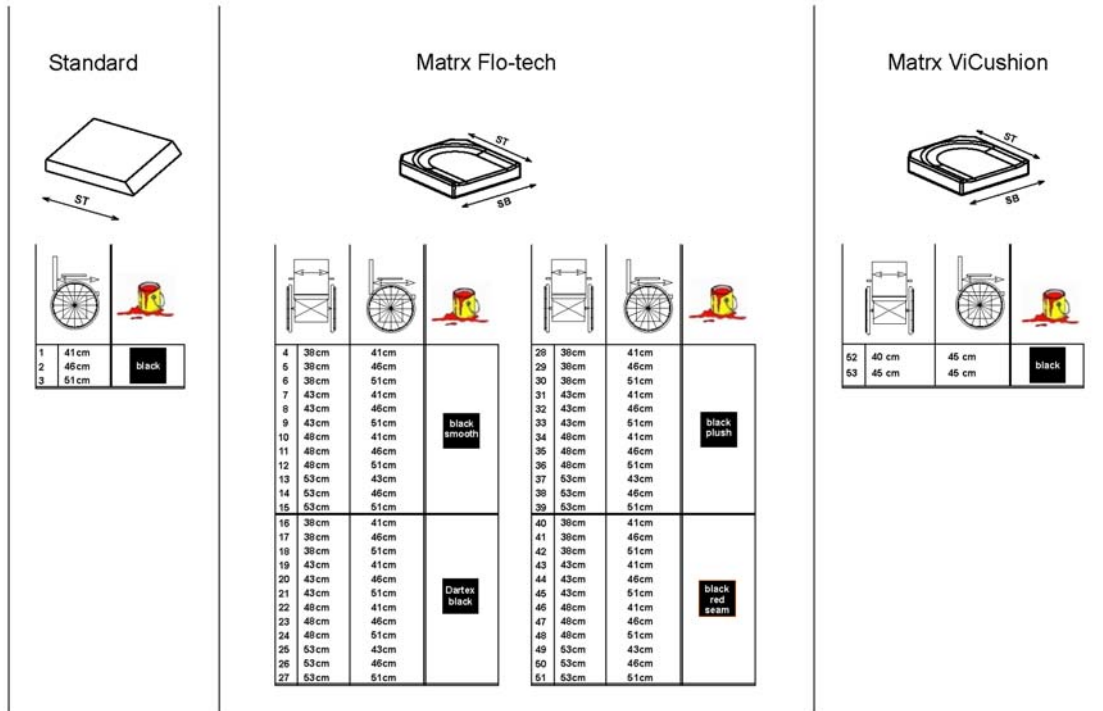

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard SD 410mm <i>[2141]</i>		1
2	1420461	Seat Cushion Standard SD 460mm <i>[2141]</i>		1
3	1420462	Seat Cushion Standard SD 510mm <i>[2141]</i>		1
4	SPFLB15/16	Flo-Tech Lite 15x16" <i>[2142+2151]</i>		1
5	SPFLB15/18	Seatcushion Easy-Adapt SB38 x ST46cm black smooth <i>[2142+2151]</i>		1
6	SPFLB15/20	Seatcushion Easy-Adapt SB38 x ST51cm black smooth <i>[2142+2151]</i>		1
7	SPFLB17/16	Seatcushion Easy-Adapt SB43 x ST41cm black smooth <i>[2142+2151]</i>		1
8	SPFLB17/18	Seatcushion Easy-Adapt SB43 x ST46cm black smooth <i>[2142+2151]</i>		1
9	SPFLB17/20	Seatcushion Easy-Adapt SB43 x ST51cm black smooth <i>[2142+2151]</i>		1
10	SPFLB19/16	Seatcushion Easy-Adapt SB48 x ST41cm black smooth <i>[2142+2151]</i>		1
11	SPFLB19/18	Seatcushion Easy-Adapt SB48 x ST46cm black smooth <i>[2142+2151]</i>		1
12	SPFLB19/20	Flo-Tech Lite 19x20" <i>[2142+2151]</i>		1
13	SPFCB21/17	Flo-Tech Contour 21x17" <i>[2142+2151]</i>		1
14	SPFCB21/18	Flo-Tech Contour 21x18" <i>[2142+2151]</i>		1
15	SPFCB21/20	Flo-Tech Contour 21x20" <i>[2142+2151]</i>		1
16	SP1529546-TR26	seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black <i>[2142+2152]</i>		1
17	SP1529547-TR26	seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black <i>[2142+2152]</i>		1
18	SP1529548-TR26	seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black <i>[2142+2152]</i>		1


Seat Cushion

Pos.	Part number	Description	Valid from → until	Qty
19	SP1529549-TR26	seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black <i>[2142+2152]</i>		1
20	SP1529550-TR26	seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black <i>[2142+2152]</i>		1
21	SP1529551-TR26	seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black <i>[2142+2152]</i>		1
22	SP1529552-TR26	seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black <i>[2142+2152]</i>		1
23	SP1529553-TR26	seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black <i>[2142+2152]</i>		1
24	SP1529554-TR26	seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black <i>[2142+2152]</i>		1
25	SP1529555-TR26	seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black <i>[2142+2152]</i>		1
26	SP1529556-TR26	seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black <i>[2142+2152]</i>		1
27	SP1529570-TR26	seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black <i>[2142+2152]</i>		1
28	SP1529546-TR35	seat cushion Flo-tech Lite SW 38 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
29	SP1529547-TR35	seat cushion Flo-tech Lite SW 38 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
30	SP1529548-TR35	seat cushion Flo-tech Lite SW 38 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
31	SP1529549-TR35	seat cushion Flo-tech Lite SW 43 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
32	SP1529550-TR35	seat cushion Flo-tech Lite SW 43 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
33	SP1529551-TR35	seat cushion Flo-tech Lite SW 43 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
34	SP1529552-TR35	Seat cushion Flo-tech Lite SW 48 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
35	SP1529553-TR35	seat cushion Flo-tech Lite SW 48 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
36	SP1529554-TR35	seat cushion Flo-tech Lite SW 48 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
37	SP1529555-TR35	seat cushion Flo-tech Contour SW 53 X SD 43 black chenille fabric <i>[2142+2154]</i>		1
38	SP1529556-TR35	seat cushion Flo-tech Contour SW 53 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
39	SP1529570-TR35	seat cushion Flo-tech Contour SW 53 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
40	SP1529546	Seatcushion Easy-Adapt SB38 x ST41cm black with red seam <i>[2142+2153]</i>		1
41	SP1529547	Seatcushion Easy-Adapt SB38 x ST46cm black with red seam <i>[2142+2153]</i>		1
42	SP1529548	Seatcushion Easy-Adapt SB38 x ST51cm black with red seam <i>[2142+2153]</i>		1
43	SP1529549	Seatcushion Easy-Adapt SB43 x ST41cm black with red seam <i>[2142+2153]</i>		1
44	SP1529550	Seatcushion Easy-Adapt SB43 x ST46cm black with red seam <i>[2142+2153]</i>		1
45	SP1529551	Seatcushion Easy-Adapt SB43 x ST51cm black with red seam <i>[2142+2153]</i>		1
46	SP1529552	Seatcushion Easy-Adapt SB48 x ST41cm black with red seam <i>[2142+2153]</i>		1
47	SP1529553	Seatcushion Easy-Adapt SB48 x ST46cm black with red seam <i>[2142+2153]</i>		1
48	SP1529554	Seatcushion Easy-Adapt SB48 x ST51cm black with red seam <i>[2142+2153]</i>		1
49	SP1529555	Seatcushion Easy-Adapt SB53 x ST43cm black with red seam <i>[2142+2153]</i>		1
50	SP1529556	Seatcushion Easy-Adapt SB53 x ST46cm black with red seam <i>[2142+2153]</i>		1

Seat Cushion

Pos.	Part number	Description	Valid from → until	Qty
51	SP1529570	Seatacushion Easy-Adapt SB53 x ST51cm black with red seam <i>[2142+2153]</i>		1
52	SP1531760	Seat Cushion MatrX Vi SW 40 x SD 45cm (16" x 18") <i>[2144]</i>		1
53	SP1531761	Seat Cushion MatrX Vi SW 45 x SD 45cm (18" x 18") <i>[2144]</i>		1
54	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm <i>[2143]</i>		1

Backrest

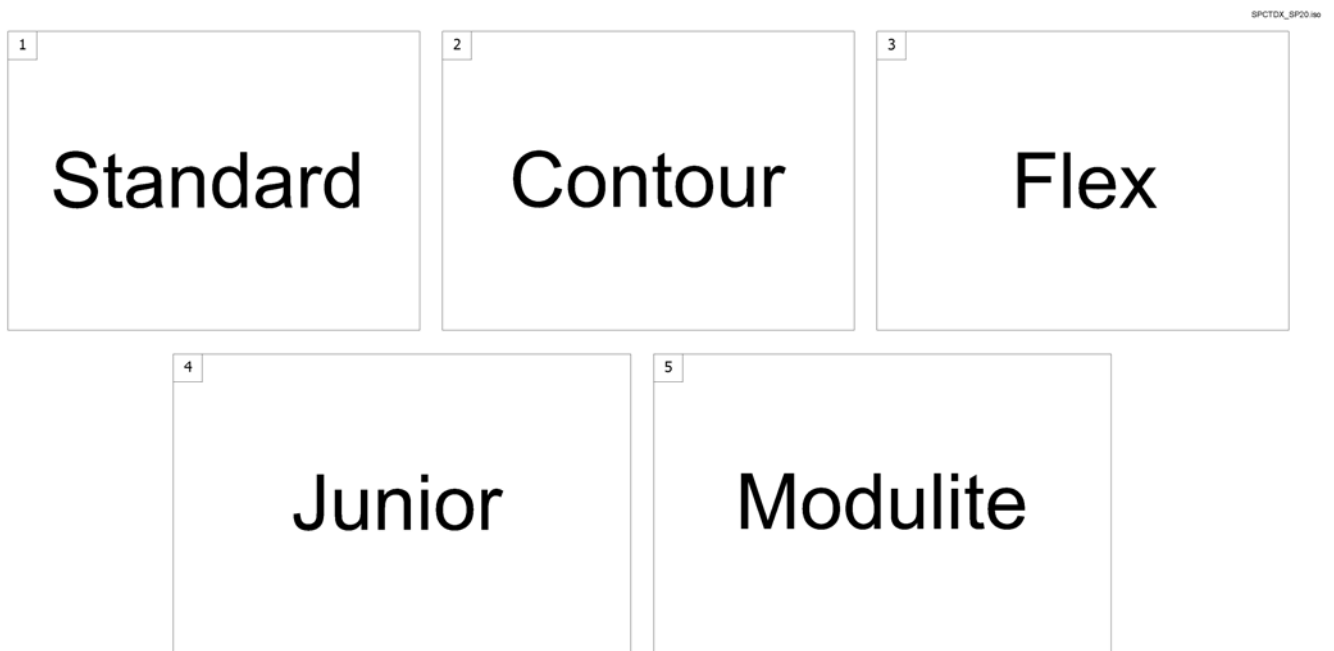


Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest for Standard Seat		1
2	-	Backrest for Contour Seat		1
3	-	Backrest for Flex Seat		1
4	-	Backrest for Junior Seat		1
5	-	Backrest for Modulite Seat		1

Backrest for Standard Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Backrest Standard Seat, settable [0958]		1
-		Backrest Recliner [0953/0954/0947]		1

Backrest Standard Seat, settable


1422061X.iso

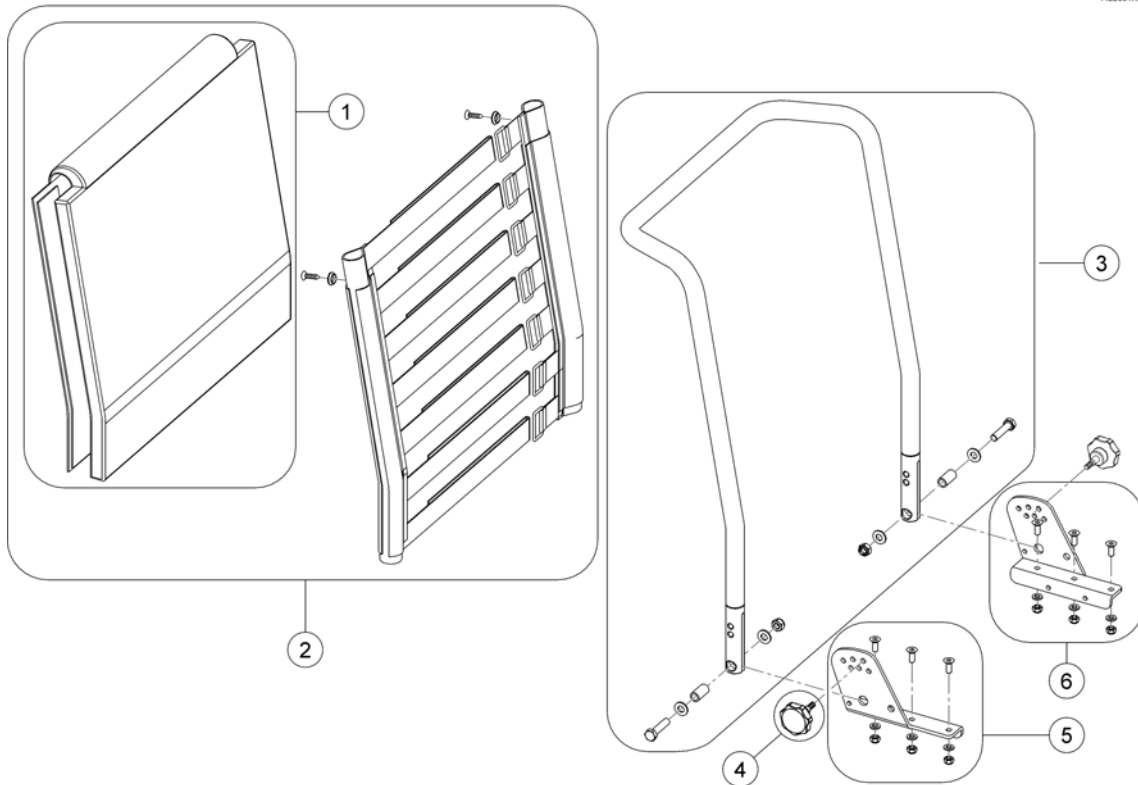

Pos.	Part number	Description	Valid from → until	Qty
1	1420445	Backrest upholstery, 480mm		1
1	1420452	Backrest upholstery, 540mm		1
2	1422062	Sling Back with Backrest upholstery 480 mm		1
2	1422063	Sling Back with Backrest upholstery 540mm		1
3	SP1419996	Backrest Frame 48cm height		1
3	SP1420465	Backrest Frame 54cm height		1
4	SP1420444	Star Grip M8		1
5	SP1541567	Backrest bracket RH		1
6	SP1541568	Backrest bracket LH		1

Backrest Recliner

1443270X.060

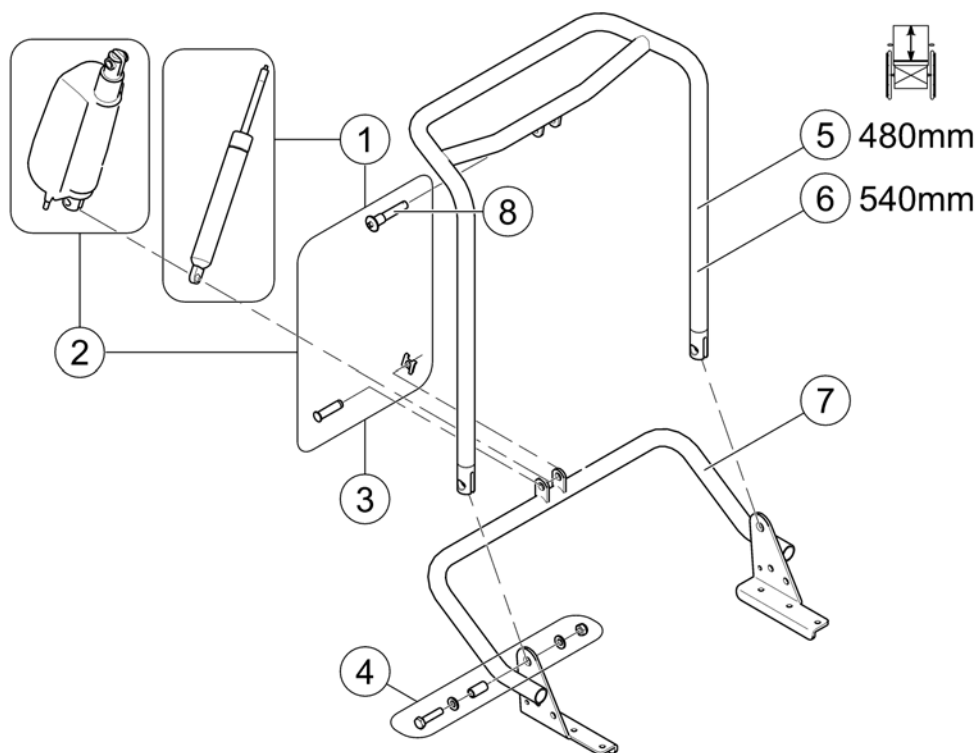


Pos.	Part number	Description	Valid from → until	Qty
-	SP1562940	Adaptercable Shark to reverse actuator		1
1	SPN368007	Gas Pressure Spring cpl.		1
2	SP1446520	Actuator ACS2 / Shark		1
3	SP1583897	Fixation Kit		1
4	SP1579749	Fixation Kit, Standard Backrest		2
5	1442864-9005	Backrest frame for 480 mm height (push handle), adjustable		1
6	1442866-9005	Backrest frame for 540 mm height (push handle), adjustable		1
7	1442845-9005	Lower Bracket for backrest frame		1
8	F368016.99	Locking Bolt - Quick Release		1

Backrest for Contour Seat


Pos.	Part number	Description	Valid from → until	Qty
-		Backrest Flex Seat		1
-		Backrest Adjustment Flex/Contour manual		1
-		Backrest adjustment Flex/Contour electric		1
-		Backrest Cushion for Contour Seat		1

Backrest Flex Seat

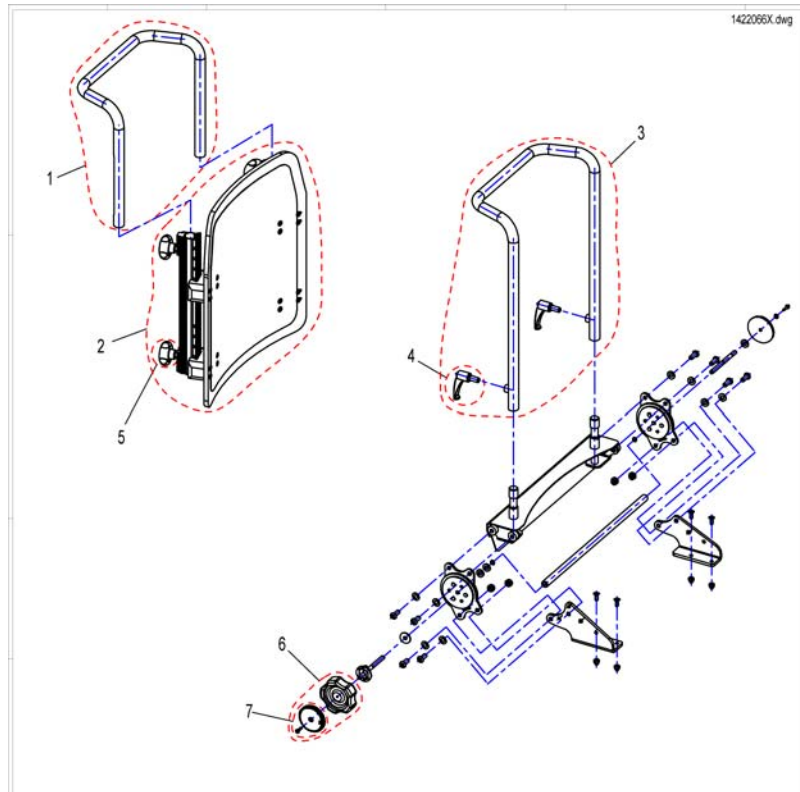

Pos.	Part number	Description	Valid from → until	Qty
1	SP1425840	Push Handle Flex SW 39cm		1
1	SP1425839	Push Handle Flex SW 43/48cm		1
2	SP1422126	Backrest Flex SW 39cm, without Upholstery, cpl.		1
2	SP1422125	Backrest Flex SW 48cm, without Upholstery, cpl.		1
2	SP1422124	Backrest Flex SW 43cm, without Upholstery, cpl.		1
3	SP1422070	Backframe Contur, Seatwidth 43/48cm, cpl.		1
4	L31996	Lever M10x20		1
5	SP5253088	Pressure Screw		1
6	SP1420483	Hand Wheel, cpl.		1
7	SP1421696	Reflector cpl.		1


Backrest Adjustment Flex/Contour manual

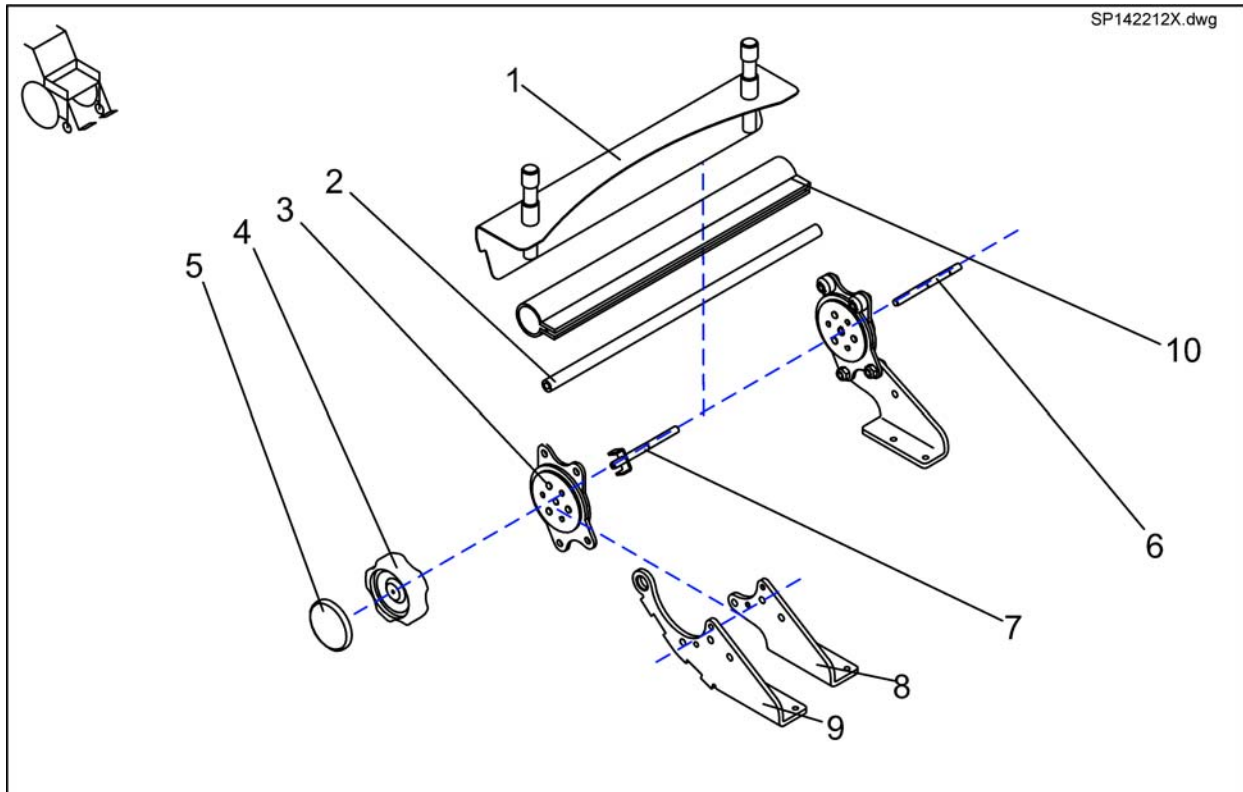

Pos.	Part number	Description	Valid from → until	Qty
-	1442942	MANUAL BACK ADJUSTER, SEAT WIDTH 43CM, FOR SLIDING ARMRESTS, CPL.		1
-	SP1422067	MANUAL BACKREST ADJUSTMENT SB 48 CPL.		1
-	SP1422122	MANUAL BACKREST ADJUSTMENT F. FLEX SB 39 CPL.		1
-	SP1425854	MANUAL BACK ADJUSTER, SEAT WIDTH 48CM, FOR SLIDING ARMRESTS, CPL.		1
-	SP1423210	BACKREST ADJUSTMENT ASSEMBLY KIT		1
-	SP1422066	MANUAL BACKREST ADJUSTMENT SB 43 CPL.		1
-	SP1425857	MANUAL BACK ADJUSTER, SEAT WIDTH 39CM, FOR SLIDING ARMRESTS, CPL.		1
1	1422112-9005	BACKREST BRACKET 480 MM		1
1	1419992-9005	BACKREST BRACKET 430 MM		1
1	1422119-9005	BACKREST BRACKET 390 MM		1
2	1420491	CONNECTING TUBE 480 MM		1
2	1420489	CONNECTING TUBE 380 MM		1
2	1420490	CONNECTING TUBE 430 MM		1
3	SP1450234	SEAT BRACE MANUAL BACKREST, 1 PAIR		1
4	1420483	HAND WHEEL		1
5	1421696	PERFORATED REFLECTOR, YELLOW		1
6	1420488-0002	SUPPORT SHAFT		1
7	1421713-0002	HAND WHEEL SHAFT		1
8	1420476-9005	BRACE BRACKET MANUAL RH		1
8	1420477-9005	BRACE BRACKET MANUAL/ELECTRICAL LH		1
9	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST		1
9	1429563-9005	FITTING ATTACHMENT RH F. SLIDING ARMREST		1
10	1442579	PROTECTIVE COVERING 480MM		1
10	1442578	PROTECTIVE COVERING 430MM		1
10	1442577	PROTECTIVE COVERING 380MM		1

Backrest adjustment Flex/Contour electric

1440456X.dwg

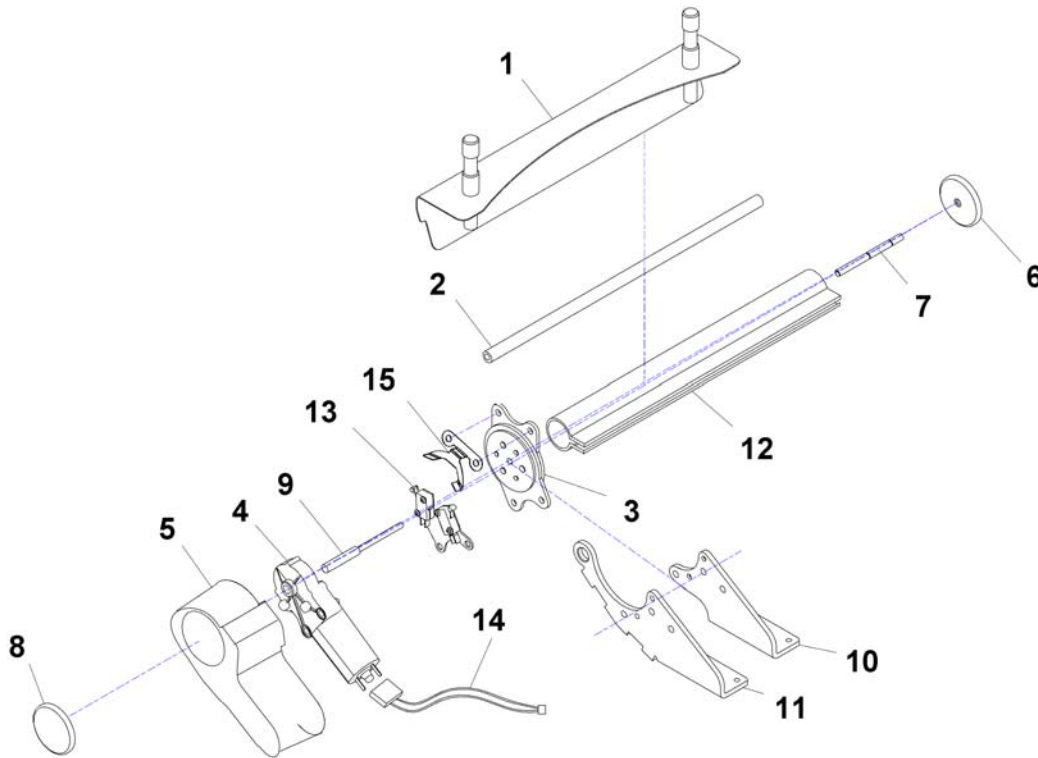

Pos.	Part number	Description	Valid from → until	Qty
	SP1440407	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 39CM, CPL.		1
	SP1440405	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 43CM, CPL.		1
	SP1440406	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 48CM, CPL.		1
	SP1440410	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 39CM, FOR SLIDING ARMRESTS, CPL.		1
	SP1440408	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 43CM, FOR SLIDING ARMRESTS, CPL.		1
	SP1440409	ELECTRICAL BACK ADJUSTER, SEAT WIDTH 48CM, FOR SLIDING ARMRESTS, CPL.		1
	SP1423210	BACKREST ADJUSTMENT ASSEMBLY KIT		1
	SP1529386	No description available.		1
1	1422119-9005	BACKREST BRACKET 390 MM		1
1	1419992-9005	BACKREST BRACKET 430 MM		1
1	1422112-9005	BACKREST BRACKET 480 MM		1
2	1420489	CONNECTING TUBE 380 MM		1
2	1420490	CONNECTING TUBE 430 MM		1
2	1420491	CONNECTING TUBE 480 MM		1
3	SP1423211	SEAT BRACE ELECTRIC BACKREST, 1 PAIR		1
4	1420475	GEAR MOTOR GMPG		1
5	1420998	MOTOR CAP		1
5	1428057	MOTOR CAP FOR ELECTRIC BACKREST SLIDING ARMREST		1
6	1421696	PERFORATED REFLECTOR, YELLOW		1
7	1420488-0002	SUPPORT SHAFT		1
8	1420482	REFLECTOR		1
9	1420487-0002	MOTOR SHAFT		1
10	1420477-9005	BRACE BRACKET MANUAL/ELECTRICAL LH		1
10	1421708-9005	BRACE BRACKET ELECTRICAL, RH		1
11	1429563-9005	FITTING ATTACHMENT RH F. SLIDING ARMREST		1
11	1429564-9005	FITTING ATTACHMENT LH F. SLIDING ARMREST		1
12	1442577	PROTECTIVE COVERING 380MM		1
12	1442578	PROTECTIVE COVERING 430MM		1
12	1442579	PROTECTIVE COVERING 480MM		1

Backrest adjustment Flex/Contour electric




Pos.	Part number	Description	Valid from → until	Qty
13	SP1440711	TWO MICRO SWITCHES WITH ASSEMBLY SHEET METAL, PRE-ASSEMBLED		1
14	SP1440726	ACTUATORCABLE RECLINER ACS2 / SHARK		1
15	1440456-0003	SWITCHING OFF SHEET METAL BACK		1

Invacare® TDX-SP, TDX-SP nB, TDX Sprint | Backrest
Backrest Cushion for Contour Seat

1573703.iso

Contour



		
1	43cm	
2	48cm	


Pos.	Part number	Description	Valid from → until	Qty
1	SP1422076	Back Cushion Contour, SW 43cm, black		1
2	SP1422078	Back Cushion Contour, SW 48cm, black		1
-	SP1423216	Mounting Hardware for Back Cushion		1

Backrest for Flex Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Backrest Flex Seat		1
-		Backrest Adjustment Flex/Contour manual		1
-		Backrest adjustment Flex/Contour electric		1
-		Backrest Cushion for Flex Seat		1

Backrest Cushion for Flex Seat

1572719 no

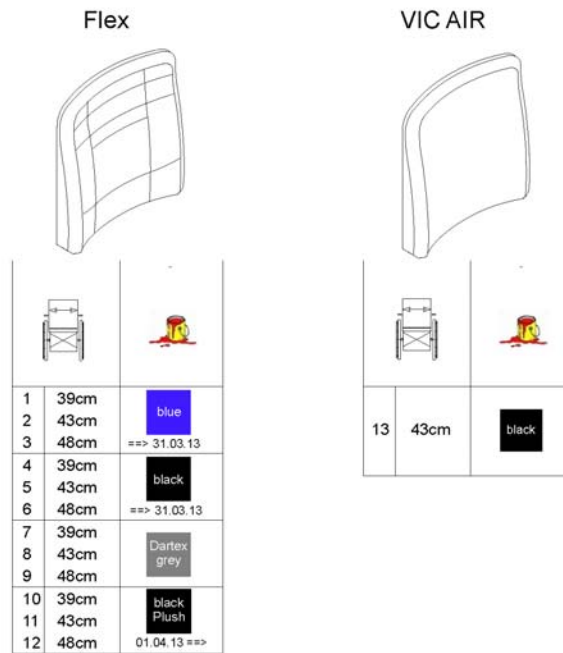


Pos.	Part number	Description	Valid from → until	Qty
1	1422534	Back Cushion SW39cm, blue	→ 2013-03-31	1
2	1422535	Back Cushion SW43cm, blue	→ 2013-03-31	1
3	1422536	Back Cushion SW48cm, blue	→ 2013-03-31	1
4	1436564	Back Cushion SW39cm, black	→ 2013-03-31	1
5	-	No longer available - please choose Black Plush	→ 2013-03-31	0
6	-	No longer available - please choose Black Plush	→ 2013-03-31	0
7	5252506-TR23	Back Cushion SW39cm, Dartex		1
8	5252507-TR23	Back Cushion SW43cm, Dartex		1
9	5252508-TR23	Back Cushion SW48cm, Dartex		1
10	SP1436564-TR35	Back Cushion SW39cm, black Plush	2013-04-01 →	1
11	SP1436565-TR35	Back Cushion SW43cm, black Plush	2013-04-01 →	1
12	SP1436566-TR35	Back Cushion SW48cm, black Plush	2013-04-01 →	1
13	1422537	Back Cushion SW43cm, VIC AIR		1

Backrest for Junior Seat


Pos.	Part number	Description	Valid from → until	Qty
-		Backrest / Junior Seat		1
-		Backrest Cushion for Junior Seat		1

Backrest / Junior Seat

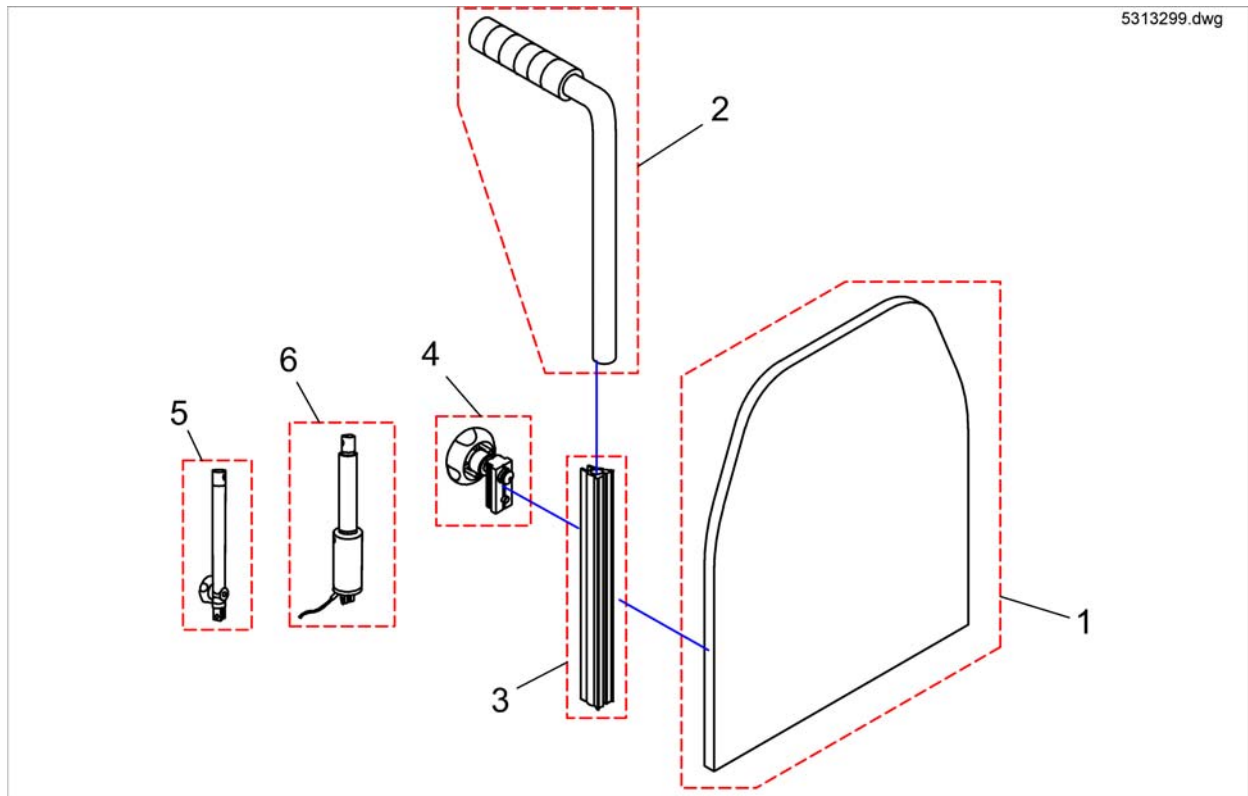

Pos.	Part number	Description	Valid from → until	Qty
1	5313162	BACKREST PLATE SB 35		1
1	5313136	BACK PLATE 30 cm		1
1	5313188	BACKREST PLATE SB 38		1
2	SP1524672	Push Handle (single one)		1
3	5252204	BACKREST BAR		1
4	5253088	Fixation block		1
5	5313220	BACKREST ADJUSTMENT MANUAL		1
5	SP1445030	MANUAL BACKREST ADJUSTMENT WITH MOUNTING KIT		1
6	SP1445029	ELECTRICAL BACKREST ADJUSTMENT WITH MOUNTING KIT		1
6	5316008	MOTOR (ACS)		1


Backrest Cushion for Junior Seat

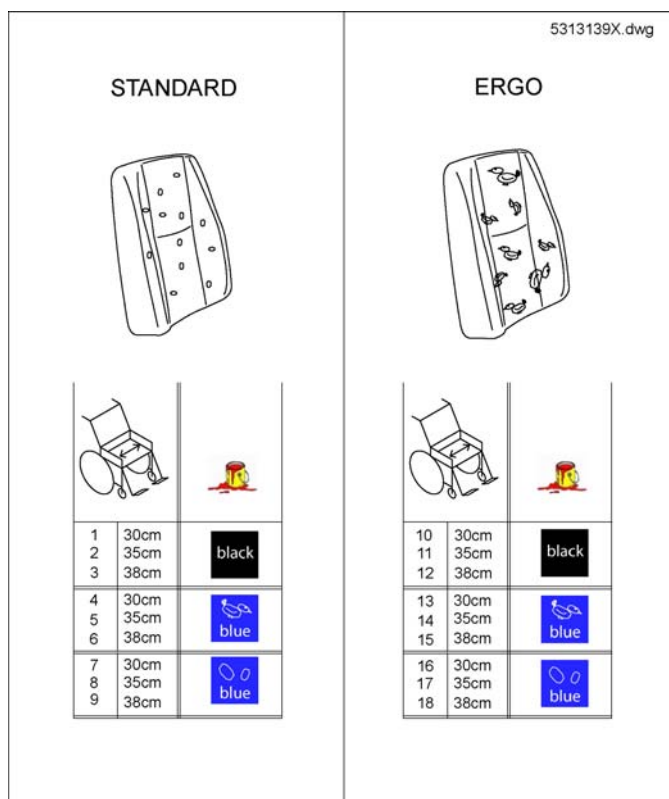

Pos.	Part number	Description	Valid from → until	Qty
1	5313142	Back Cushion STD SW30cm, Vinyl Black		1
2	5313168	Back Cushion STD SW35cm, Vinyl Black		1
3	5313194	Back Cushion STD SW38cm, Vinyl Black		1
4	5313139	Back Cushion STD SW30cm, Blue Birdie		1
5	5313165	Back Cushion STD SW35cm, Blue Birdie		1
6	5313191	Back Cushion STD SW38cm, Blue Birdie		1
7	5313141	Back Cushion STD SW30cm, Blue Dot		1
8	5313167	Back Cushion STD SW35cm, Blue Dot		1
9	5313193	Back Cushion STD SW38cm, Blue Dot		1
10	5313146	Back Cushion Ergo SW30cm, Vinyl Black		1
11	5313172	Back Cushion Ergo SW35cm, Vinyl Black		1
12	5313198	Back Cushion Ergo SW38cm, Vinyl Black		1
14	5313169	Back Cushion Ergo SW35cm, Blue Birdie		1
15	5313195	Back Cushion Ergo SW38cm, Blue Birdie		1
16	5313145	Back Cushion Ergo SW30cm, Blue Dot		1
17	5313171	Back Cushion Ergo SW35cm, Blue Dot		1
18	5313197	Back Cushion Ergo SW38cm, Blue Dot		1

Backrest for Modulite Seat


1586095.000

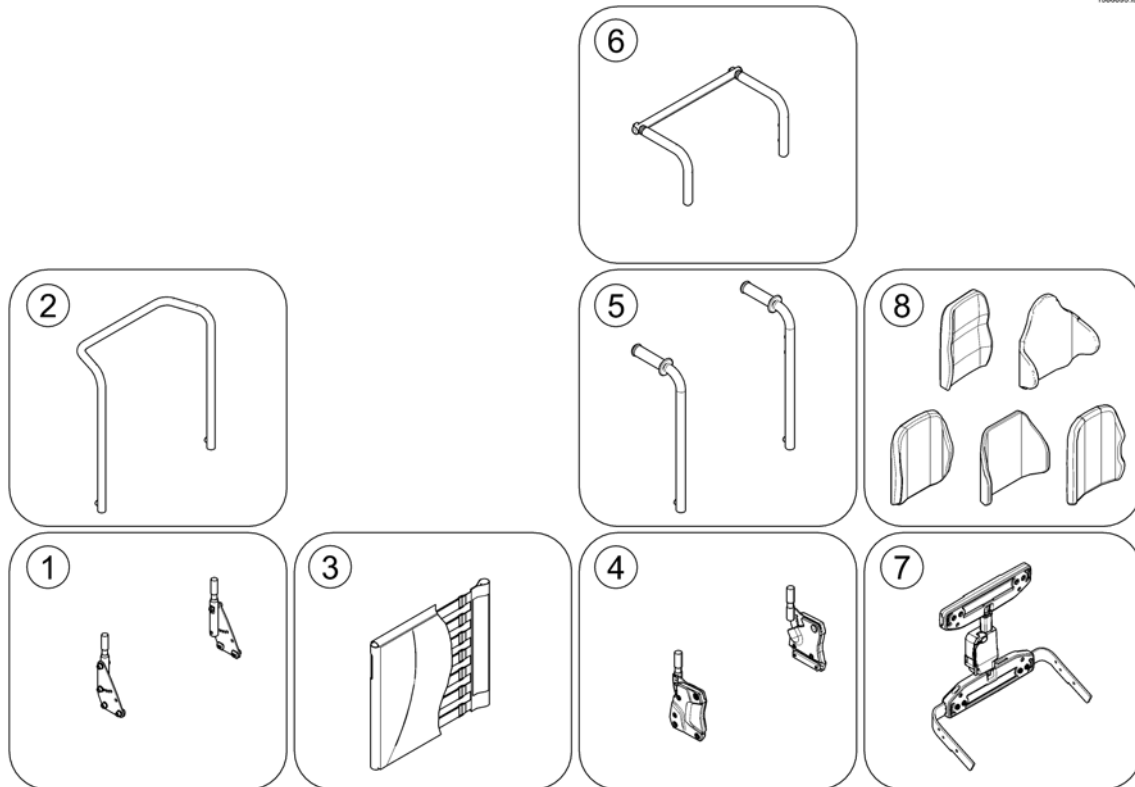

Pos.	Part number	Description	Valid from → until	Qty
1	-	Support for Backrest Frame <i>[0975+2161]</i>		0
2	-	Backrest Frame <i>[2161]</i>		0
3	-	Sling Back, tension adjustable <i>[2171]</i>		0
4	-	Backrest support		0
5	-	Backposts <i>[2162/2163]</i>		0
6	-	Connection tube <i>[1307]</i>		0
7	-	Back Recliner / Stabilization bracket <i>[0977]</i>		0
8	-	Backrest Cushions <i>[2172/2173/2174/2175/2176/2177/2178]</i>		1

Support for Backrest Frame


1571553 no

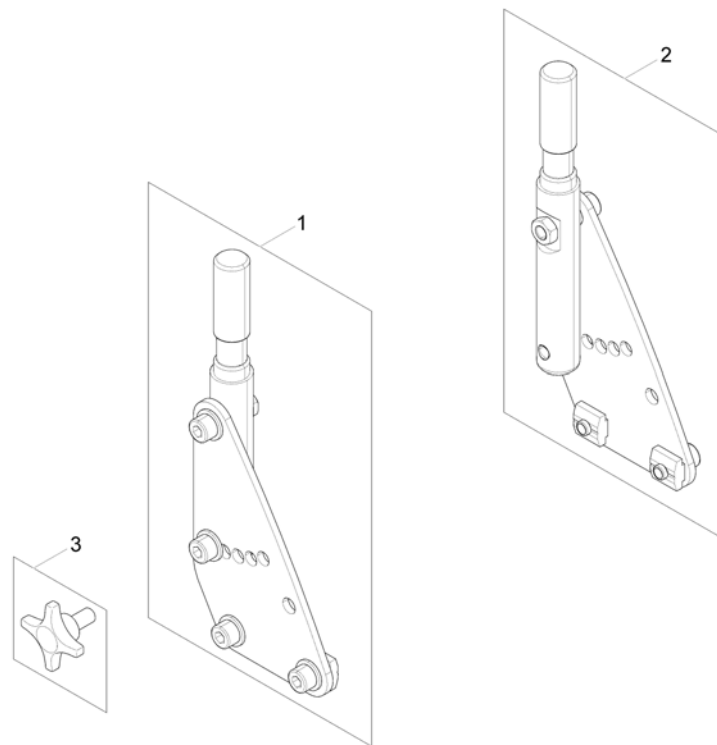

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571542	Backrest Support RH		1
2	SP1571553	Backrest Support LH		1
3	5824117	hand wheel M8x15 <i>(only swedish homologation)</i>		2

Backrest Frame


1571831.iss

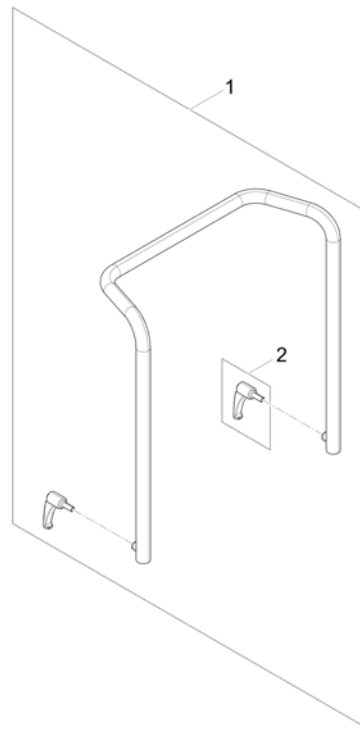

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567810	Backrest Frame		1
2	0021-70002	Indexable lever 63mm M8x16		1

Sling Back, tension adjustable


1571829.000

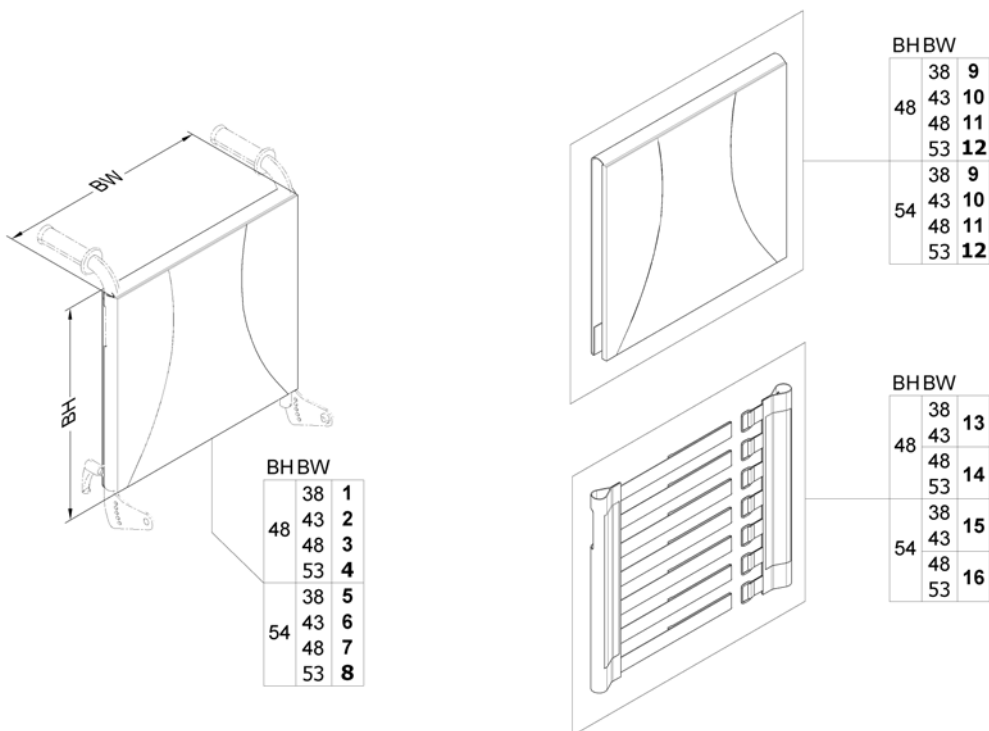

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567529	Sling Back, tension adjustable & Upholstery, BW 380mm / BH 480mm, cpl.		1
2	SP1567465	Sling Back, tension adjustable & upholstery, BW 430mm / BH 480mm, cpl.		1
3	SP1567530	Sling Back, tension adjustable & upholstery, BW 480mm / BH 480mm, cpl.		1
4	SP1567531	Sling Back, tension adjustable & upholstery, BW 530mm / BH 480mm, cpl.		1
5	SP1567627	Sling Back, tension adjustable & upholstery, BW 380mm / BH 540mm, cpl.		1
6	SP1567746	Sling Back, tension adjustable & upholstery, BW 430mm / BH 540mm, cpl.		1
7	SP1567747	Sling Back, tension adjustable & upholstery, BW 480mm / BH 540mm, cpl.		1
8	SP1567748	Sling Back, tension adjustable & upholstery, BW 530mm / BH 540mm, cpl.		1
9	SP1563563	Backrest upholstery		1
10	SP1563653	Backrest upholstery		1
11	SP1563654	Backrest upholstery		1
12	SP1563655	Backrest upholstery		1
13	SP1571492	Sling back		1
14	SP1571493	Sling back		1
15	SP1571494	Sling back		1
16	SP1571510	Sling back		1


Backrest support

1572928 Rev

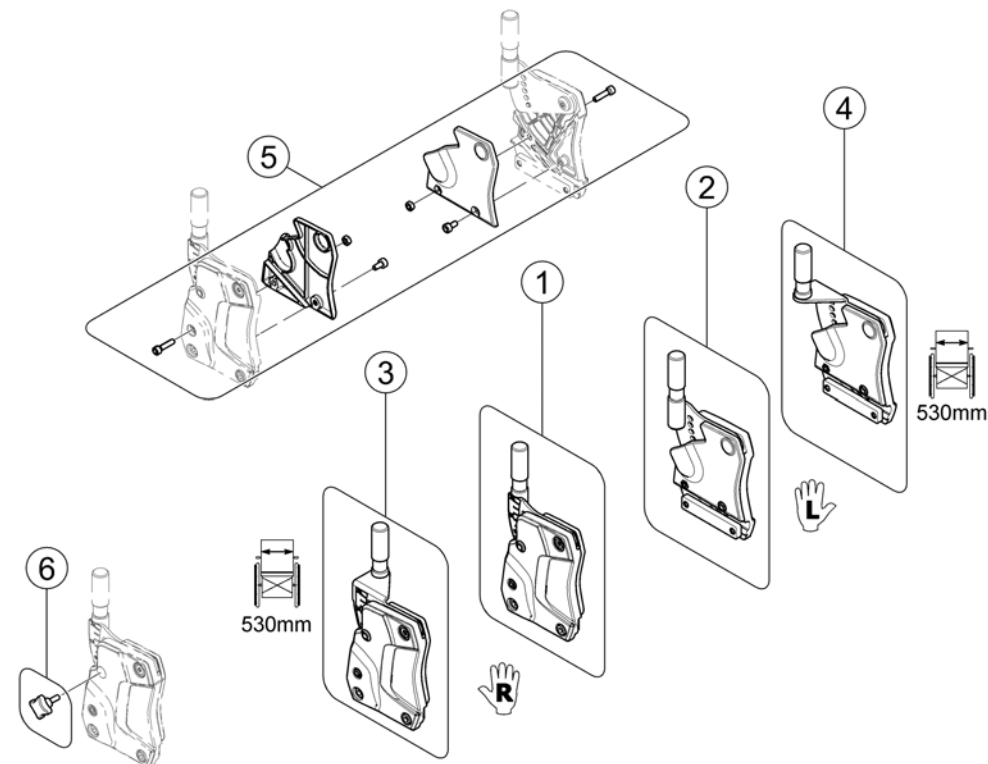

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582709	Backrest Support - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571560</i>		1
2	SP1582708	Backrest Support - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571561</i>		1
3	SP1582952	Backrest Support SW 530mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571568</i>		1
4	SP1582951	Backrest Support SW 530mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1571569</i>		1
5	SP1571676	Cover		1
6	SP1579877	Hand Screw M6 <i>(Swedish Homologation only!)</i>	→ 2015-07-31	2
6	SP1589324	Cross handle M8, cpl. <i>(Swedish Homologation only!)</i>	2015-08-01 →	2


Backposts

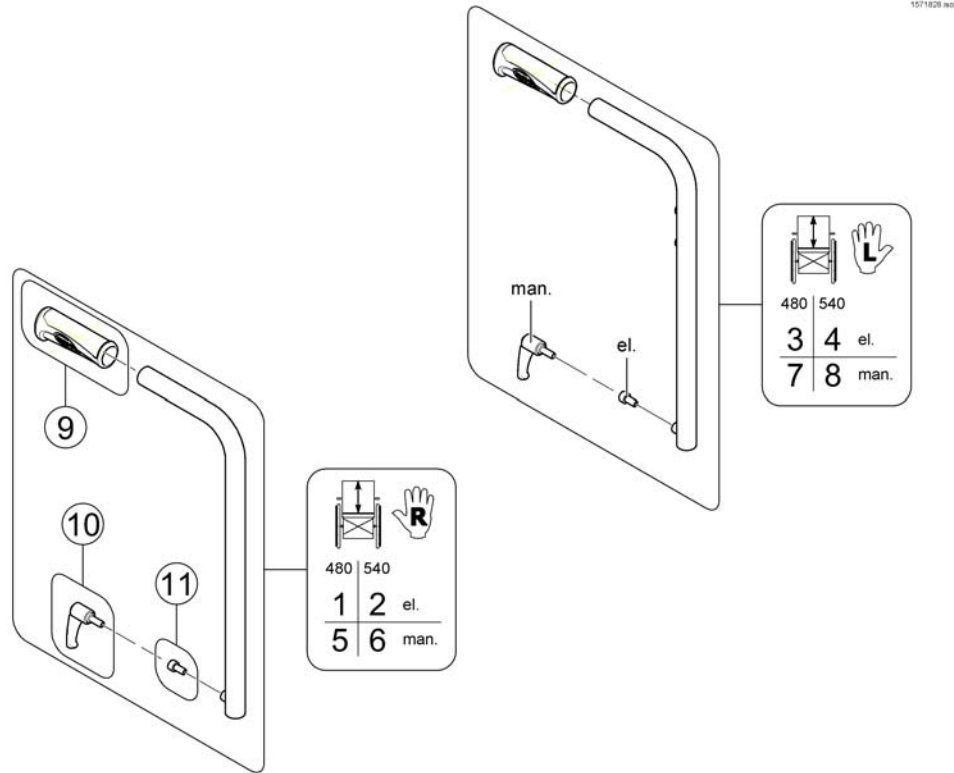

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587177	Backpost el. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563102</i>		1
2	SP1587184	Backpost el. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1563103</i>		1
3	SP1587176	Backpost el. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1560576</i>		1
4	SP1587183	Backpost el. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1559578</i>		1
5	SP1587163	Backpost man. BH 480mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572489</i>		1
6	SP1587182	Backpost man. BH 540mm - RH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572490</i>		1
7	SP1587162	Backpost man. BH 480mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572491</i>		1
8	SP1587178	Backpost man. BH 540mm - LH <i>Compatible with Flip-Up & Following Armrest [2193/2192/2194] - replaces SP1572492</i>		1
9	SP1586122	Handles (pair) <i>replaces item SP1563651</i>		1
10	0021-70002	Indexable lever 63mm M8x16		1
11	1567999	Bolt M8x16 8.8 black DIN912		1

Connection tube


1571835 iso

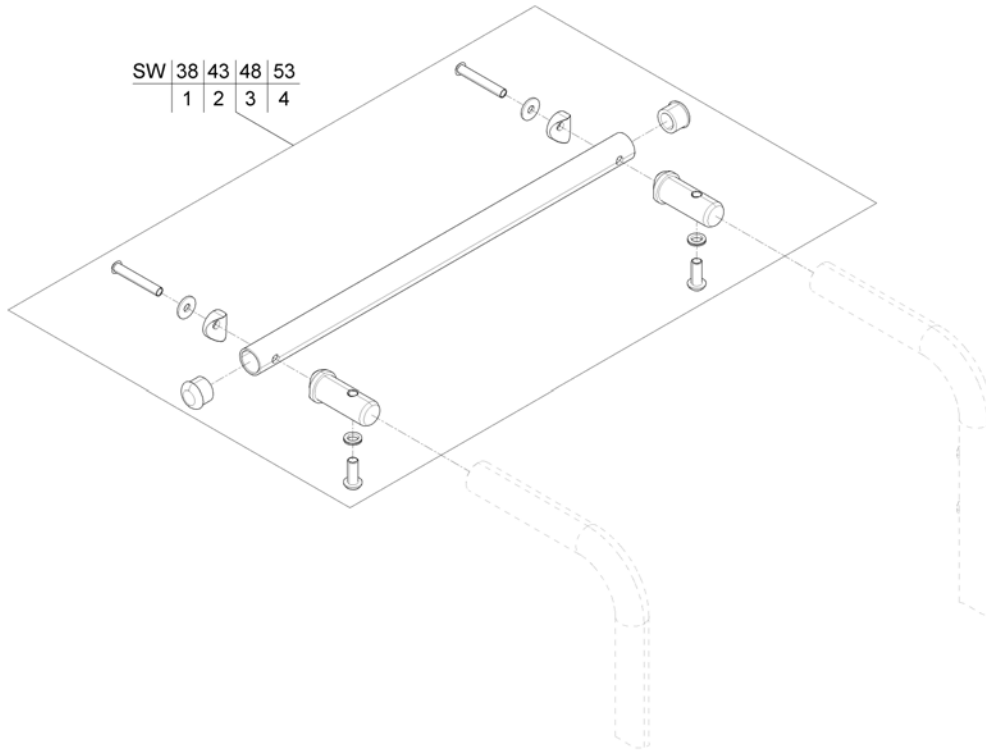

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570370	Connection tube		1
2	SP1570371	Connection tube		1
3	SP1570372	Connection tube		1
4	SP1570373	Connection tube		1

Back Recliner / Stabilization bracket


1571007.kso

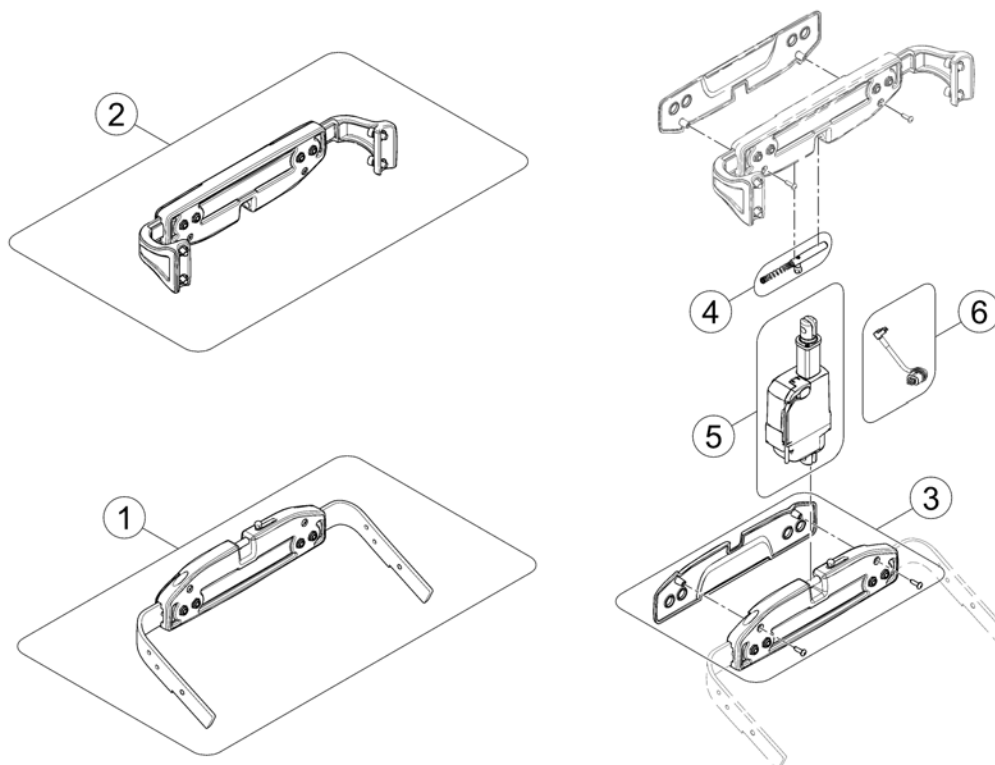

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567169	Actuator bracket (pre set to SW 43cm)		1
2	SP1567168	Stabilization bracket (pre set to SW 43cm)		1
3	SP1571702	Cover		1
4	SP1571704	Locking bolt		1
5	SP1557820	Actuator		1
6	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1

Backrest Cushions


1588915.iss

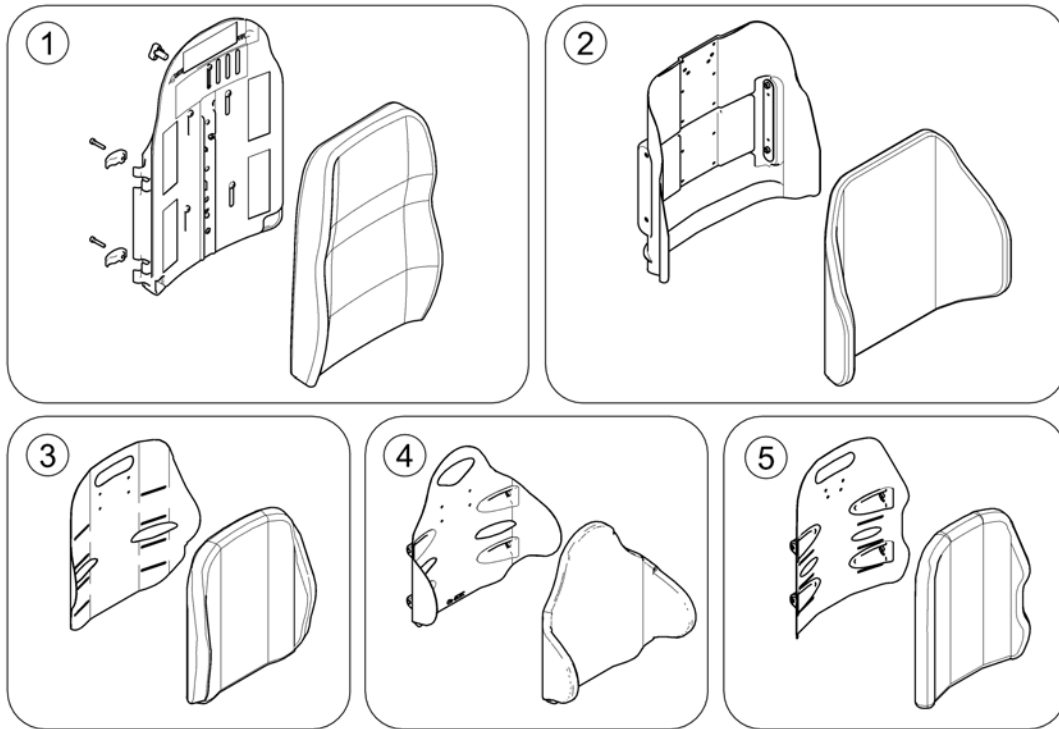

Pos.	Part number	Description	Valid from → until	Qty
1	-	Flex 3 / Laguna		1
2	-	Matrx Posture Back		1
3	-	Matrx Elite T/R Back	2015-07-01 →	1
4	-	Matrx Elite Deep Back	2015-07-01 →	1
5	-	Matrx Elite (Full) Back		1


Flex 3 / Laguna

1573106.mw

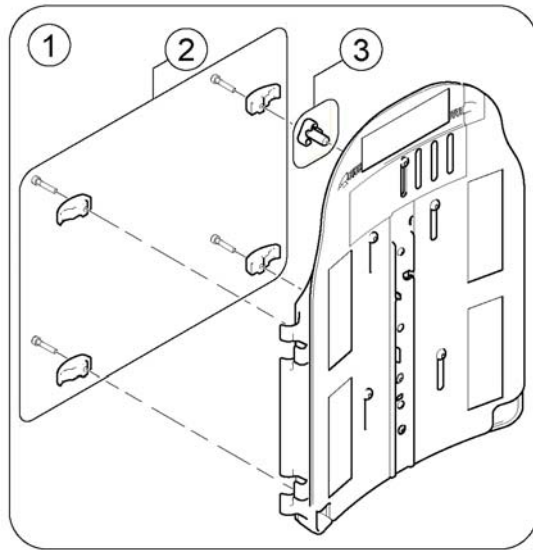

Flex 3

Laguna

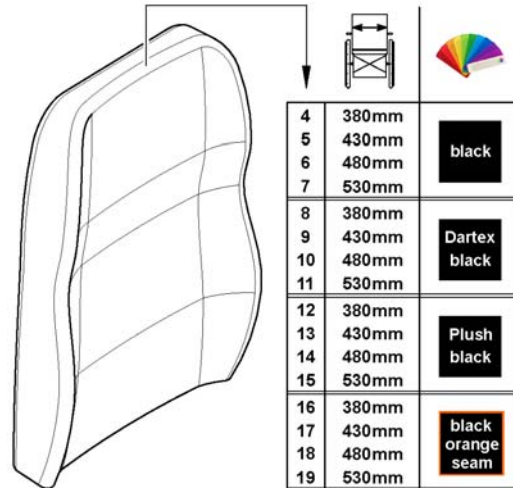

Pos.	Part number	Description	Valid from → until	Qty
1	SP1568489	Flex3 backrest plate <i>[2172]</i>		1
2	SP1572321	Clamps		1
3	1497411	HANDWHEEL M12 LONG NECK/HEADREST		1
4	SP1529585	Backrest Cushion Laguna / EasyAdapt, black - SW 380mm <i>[2173+2151]</i>		1
5	SP1529586	Backrest Cushion Laguna / EasyAdapt, black - SW 430mm <i>[2173+2151]</i>		1
6	SP1529587	Backrest Cushion Laguna / EasyAdapt, black - SW 480mm <i>[2173+2151]</i>		1
7	SP1529596	Backrest Cushion Laguna / EasyAdapt, black - SW 530mm <i>[2173+2151]</i>		1
8	SP1529538-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 380mm <i>[2173+2152]</i>		1
9	SP1529539-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 430mm <i>[2173+2152]</i>		1
10	SP1529540-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 480mm <i>[2173+2152]</i>		1
11	SP1529545-TR26	Backrest Cushion Laguna / EasyAdapt, Dartex black - SW 530mm <i>[2173+2152]</i>		1
12	SP1529538-TR35	Backrest Cushion Laguna, Plush black - SW 380mm <i>[2173+2154]</i>		1
13	SP1529539-TR35	Backrest Cushion Laguna, Plush black - SW 430mm <i>[2173+2154]</i>		1
14	SP1529540-TR35	Backrest Cushion Laguna, Plush black - SW 480mm <i>[2173+2154]</i>		1
15	SP1529545-TR35	Backrest Cushion Laguna, Plush black - SW 530mm <i>[2173+2154]</i>		1
16	SP1529538	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 380mm <i>[2173+2153]</i>		1
17	SP1529539	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 430mm <i>[2173+2153]</i>		1
18	SP1529540	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 480mm <i>[2173+2153]</i>		1


Flex 3 / Laguna

Pos.	Part number	Description	Valid from → until	Qty
19	SP1529545	Backrest Cushion Laguna / EasyAdapt, black with orange seam - SW 530mm <i>[2173+2153]</i>		1

Matrx Posture Back

158885 Rev

Posture Back

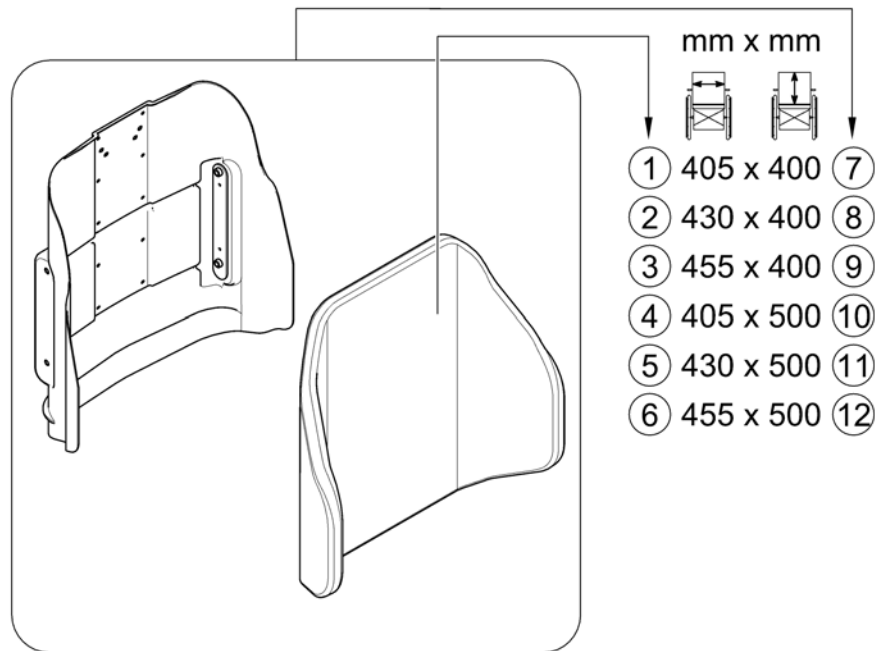


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRS0139-1616	Backrest Cushion - Posture Back 405x400mm (16x16")		1
2	SPTRS0139-1716	Backrest Cushion - Posture Back 430x400mm (17x16")		1
3	SPTRS0139-1816	Backrest Cushion - Posture Back 455x400mm (18x16")		1
4	1544897	Backrest Cushion - Posture Back 405x500mm (16x20")		1
5	SPTRS0139-1720	Backrest Cushion - Posture Back 430x500mm (17x20")		1
6	SPTRS0139-1820	Backrest Cushion - Posture Back 455x500mm (18x20")		1
7	SP1531762	Backrest, Matrx Posture Back 16x16", cpl. <i>[2174]</i>		1
8	SP1531792	Backrest, Matrx Posture Back 17x16", cpl. <i>[2174]</i>		1
9	SP1531822	Backrest, Matrx Posture Back 18x16", cpl. <i>[2174]</i>		1
10	SP1531763	Backrest, Matrx Posture Back 16x20", cpl. <i>[2175]</i>		1
11	SP1531793	Backrest, Matrx Posture Back 17x20", cpl. <i>[2175]</i>		1
12	SP1531823	Backrest, Matrx Posture Back 18x20", cpl. <i>[2175]</i>		1

Matrx Elite T/R Back

1588888 no

Elite T/R Back

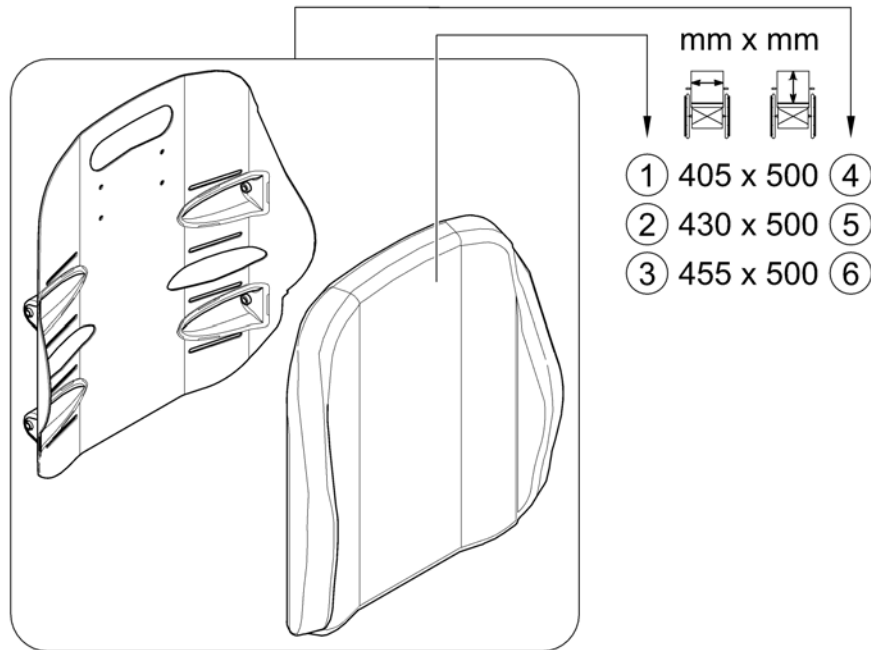


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRS0375-1620	Backrest Cushion - Elite T/R Back 405x500mm (16x20")		1
2	SPTRS0375-1720	Backrest Cushion - Elite T/R Back 430x500mm (17x20")		1
3	SPTRS0375-1820	Backrest Cushion - Elite T/R Back 455x500mm (18x20")		1
4	SP1588270	Backrest, Matrx Elite T/R Back 16x20", cpl. [2177+2121]		1
5	SP1588271	Backrest, Matrx Elite T/R Back 17x20", cpl. [2177+2122]		1
6	SP1588272	Backrest, Matrx Elite T/R Back 18x20", cpl. [2177+2123]		1

Matrx Elite Deep Back

1588713 Rev

Elite Deep Back

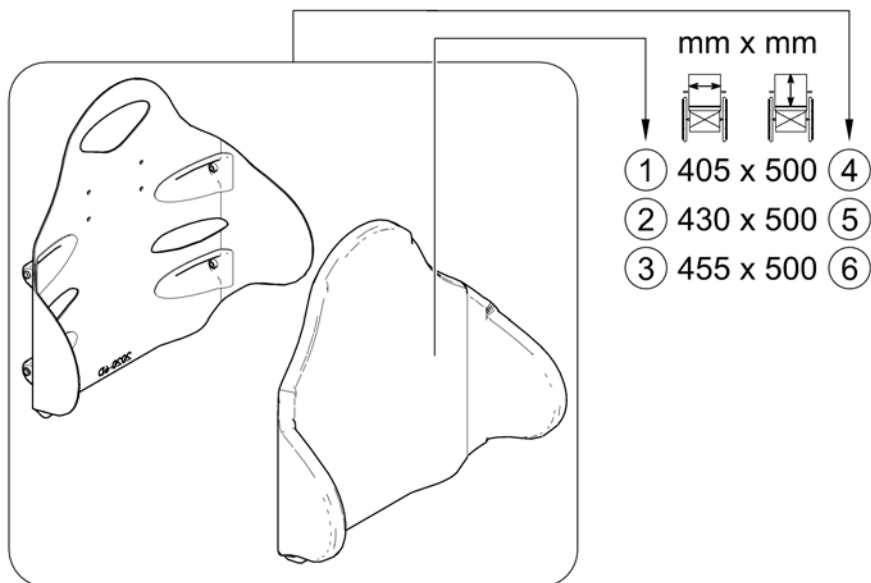


Pos.	Part number	Description	Valid from → until	Qty
1	SPTRS0344-1620	Backrest Cushion - Elite Deep Back 405x500mm (16x20")		1
2	SPTRS0344-1720	Backrest Cushion - Elite Deep Back 430x500mm (17x20")		1
3	SPTRS0344-1820	Backrest Cushion - Elite Deep Back 455x500mm (18x20")		1
4	SP1588273	Backrest, Matrx Elite Deep Back 16x20", cpl. [2178+2121]		1
5	SP1588274	Backrest, Matrx Elite Deep Back 17x20", cpl. [2178+2122]		1
6	SP1588275	Backrest, Matrx Elite Deep Back 18x20", cpl. [2178+2123]		1

Matrx Elite (Full) Back


1588714.00

Elite Full Back

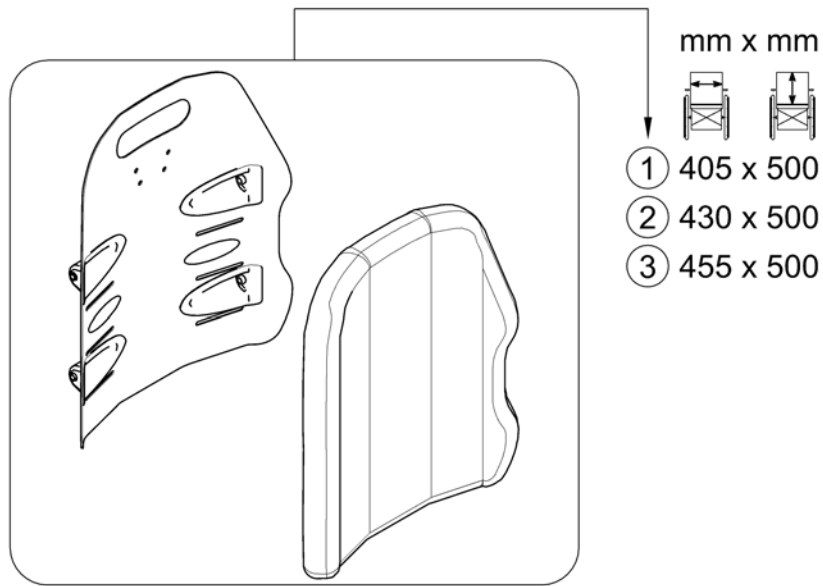

Pos.	Part number	Description	Valid from → until	Qty
1	SP1531824	Backrest, Matrx Elite (Full) Back 16x20", cpl. [2176]		1
2	SP1531831	Backrest, Matrx Elite (Full) Back 17x20", cpl. [2176]		1
3	SP1531832	Backrest, Matrx Elite (Full) Back 18x20", cpl. [2176]		1


Legrest fixation

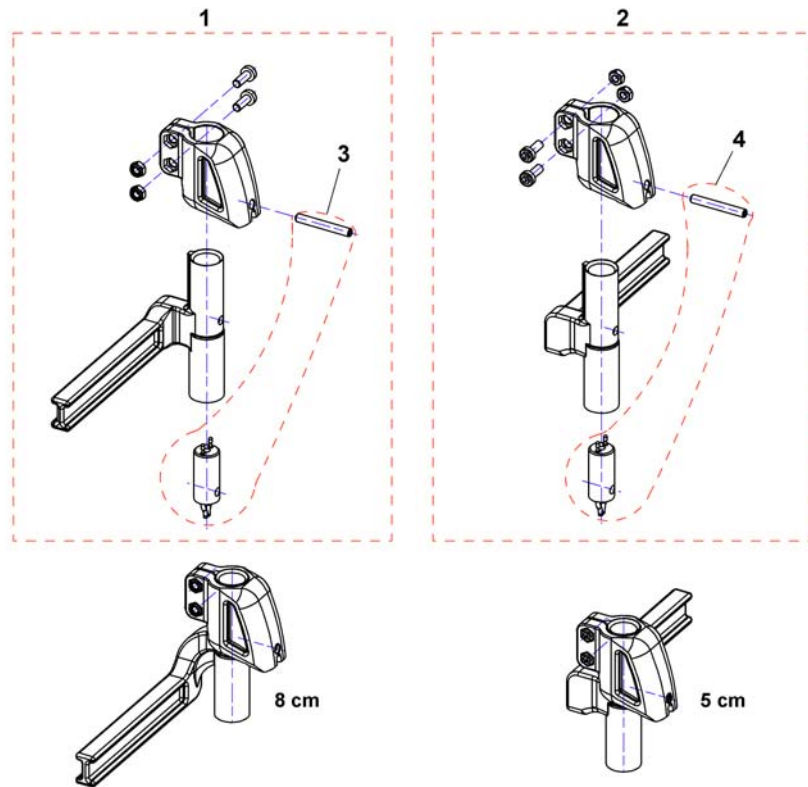

1562243.dwg

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449733	Legrest Support Vari series, man. 5cm - LH		1
1	SP1449737	Footrest/Legrest receptacle, manuell, LH, 8cm		1
1	SP1449735	Footrest/Legrest receptacle, powered, LH, 5cm		1
1	SP1449739	Footrest/Legrest receptacle, powered, LH, 8cm		1
1	SP1529951	Footrest/Legrest receptacle, powered, LH, 5cm <i>For TDX-SP with lifter</i>		1
1	SP1535937	Footrest/Legrest receptacle, powered, LH, 8cm <i>For TDX-SP with lifter</i>		1
2	SP1449732	Legrest Support Vari series, man. 5cm - RH		1
2	SP1449736	Footrest/Legrest receptacle, manuell, RH, 8cm		1
2	SP1449314	Footrest/Legrest receptacle, powered, RH, 5cm		1
2	SP1449315	Footrest/Legrest receptacle, powered, RH, 8cm		1
2	SP1529949	Footrest/Legrest receptacle, powered, RH, 5cm <i>For TDX-SP with lifter</i>		1
2	SP1535856	Footrest/Legrest receptacle, powered, RH, 8cm <i>For TDX-SP with lifter</i>		1
3	SP1450228	Power Supply Cable Legrest (990mm)		1
3	SP1527775	Power Supply Cable Legrest (650mm) <i>For TDX-SP with lifter</i>		1
4	SP1450232	Power Supply Cable Legrest (550mm)		1
4	SP1527775	Power Supply Cable Legrest (650mm) <i>For TDX-SP with lifter</i>		1
-	SP1527899	Extension Cable for powered Legrest <i>For TDX-SP without lifter</i>		1


Hanger Vari A (04/2009 =>)

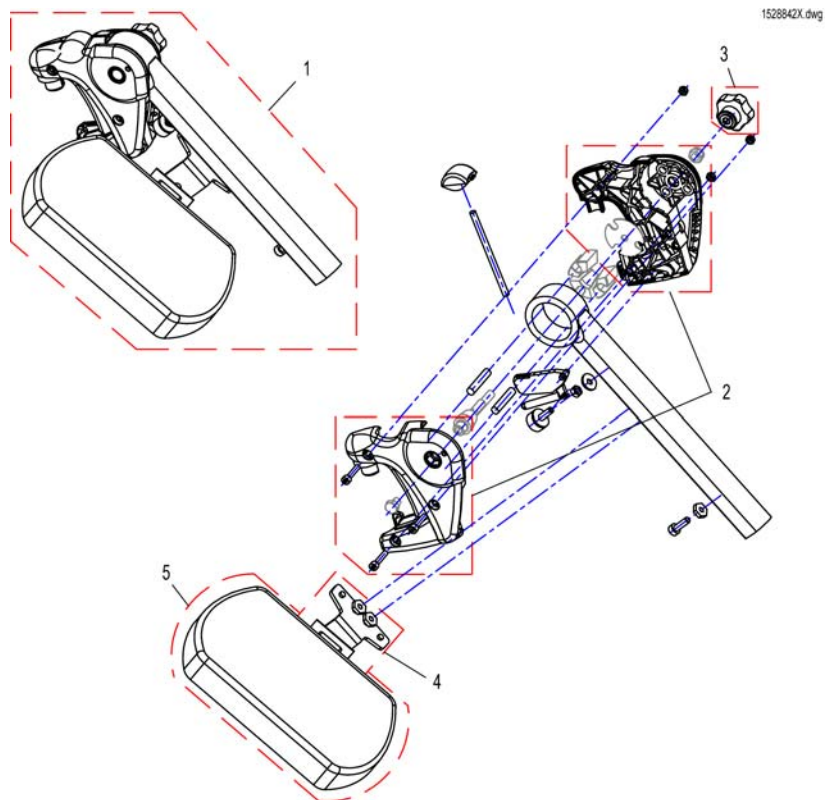

Pos.	Part number	Description	Valid from → until	Qty
-	SP1527820	Rework-Kit Vari A Legrest Version 1 => Version 2 <i>Vari A Version 1 => Version 2 Upgrade Kit</i>		1
1	SP1527498	Hanger Vari A LH, cpl.		1
1	SP1527492	Hanger Vari A RH, cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	L34011	Handknob M8 Tristarfemale		1
4	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
4	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
5	6301333	Legrest calf pad		1

Hanger Vari F


144934 / X.dwg

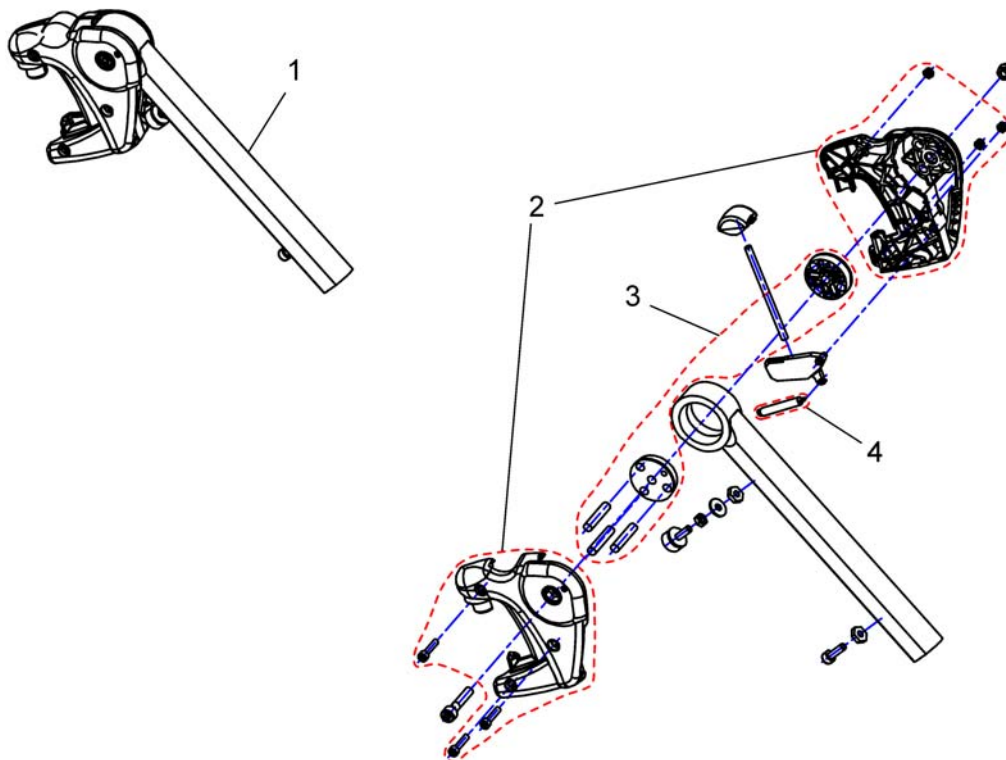

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449347	Hanger Vari F LH cpl.		1
1	SP1449346	Hanger Vari F RH cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	1434956	Mechanic		1
4	SPZ173178	Spring		1

Hangers ADM LH/RH manual


1449409X.dwg

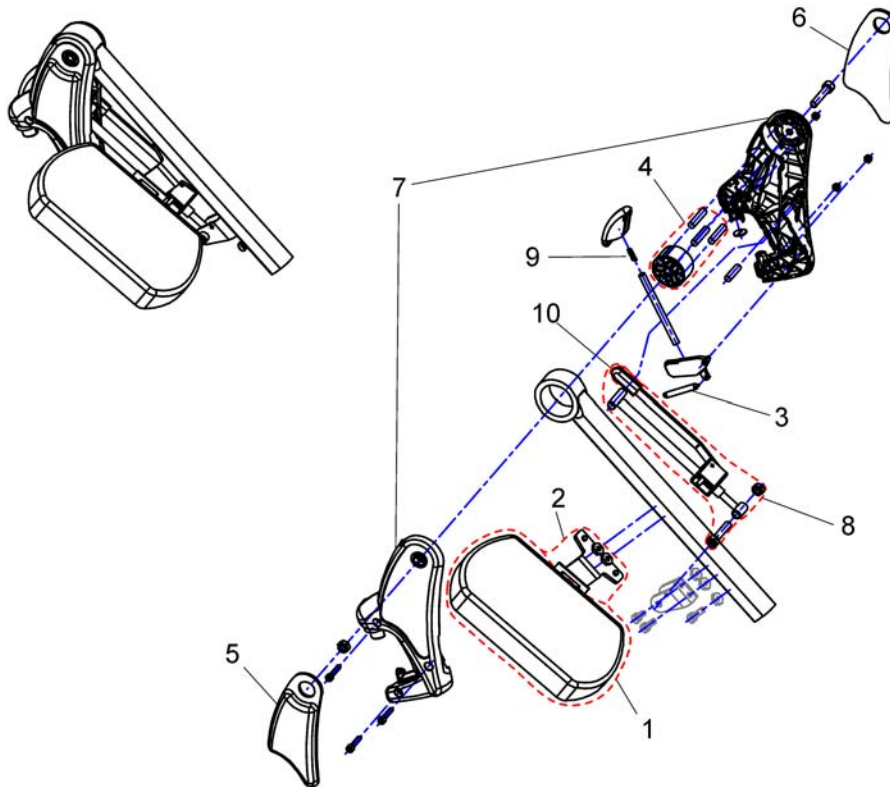

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450410	Knee Upholstery LH		1
6	SP1450448	Knee Upholstery RH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ174130	ADM Adjustment mechanism cpl.		1
9	SPN12196	Spring		1
10	SP1006490	Cap for tripping lever		1
11	SP1449410	Hanger ADM RH cpl.		1
11	SP1449409	Hanger ADM LH cpl.		1

Hanger Vari ADE/ADP electric (07/2005 =>)


1449407X.dwg

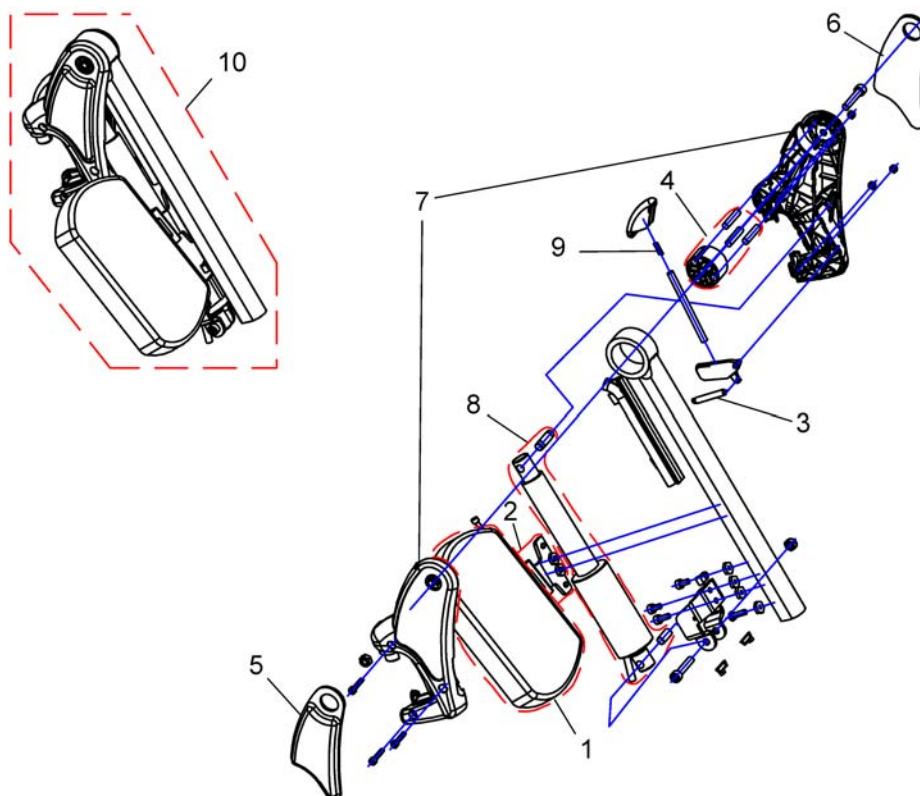

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450448	Knee Upholstery RH		1
6	SP1450410	Knee Upholstery LH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ151006	Actuator with circuit board, preassembled		1
9	SPN12196	Spring		1
10	SP1449408	Hanger ADP RH cpl.		1
10	SP1449407	Hanger ADP LH cpl.		1


Central Legrest, manual - Standard- / Easy Adapt-Seat

1537926X.iso

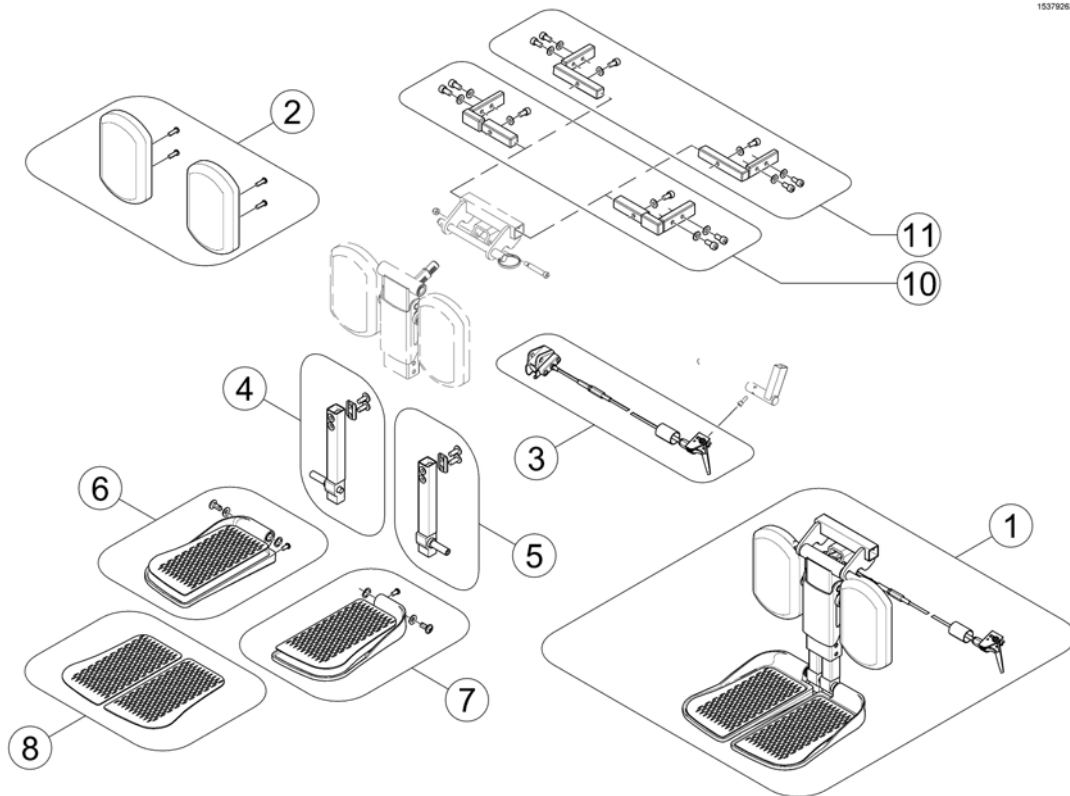

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537926	Central Legrest manual, cpl.		1
2	SP1163373	Calfplates (Pair)		1
3	SP1121079	Bowden cable central legrest, manual		1
4	SP1161986	Kit, extension tube, cpl. RH		1
5	SP1161987	Kit, extension tube, cpl. LH		1
6	SP1161990	Footplate, cpl. RH		1
7	SP1161991	Footplate, cpl. LH		1
8	SP1163375	Footrest pad (Pair)		1
10	SP1541397	Support center mounted legrest 430-530 mm		1
11	SP1541398	Support center mounted legrest 380-410 mm		1


Central Legrest, manual - Modulite Seat

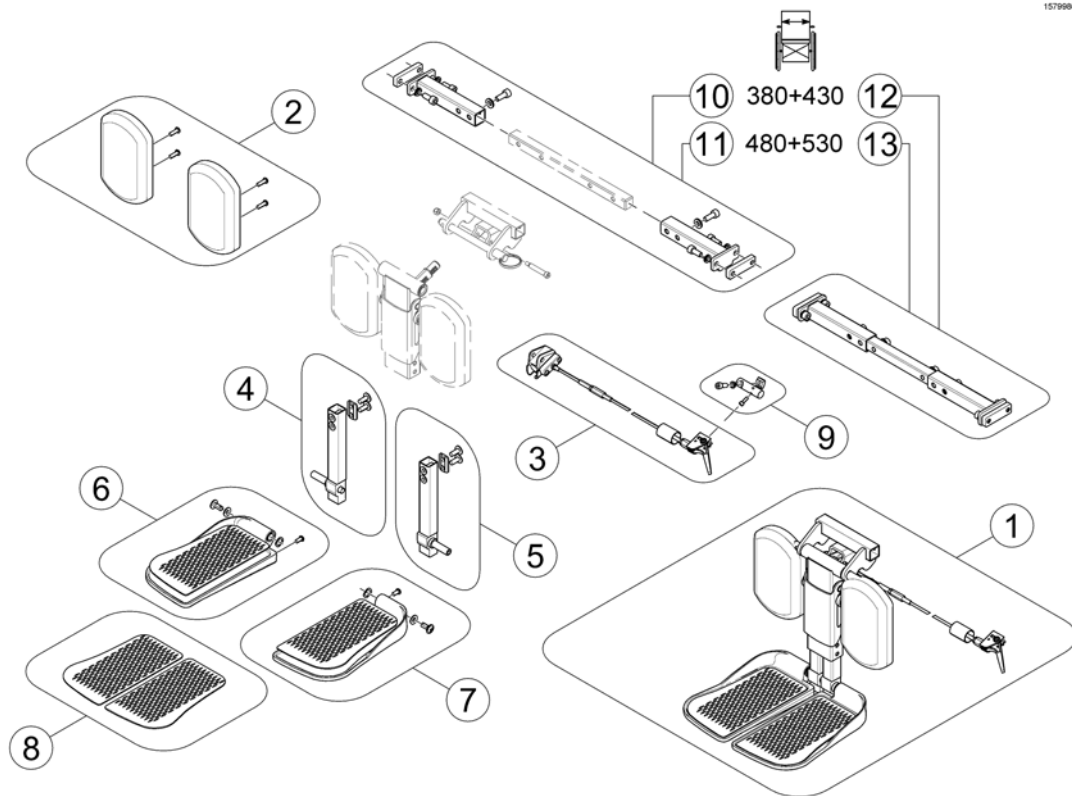

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537926	Central Legrest manual, cpl.		1
2	SP1163373	Calfplates (Pair)		1
3	SP1121079	Bowden cable central legrest, manual		1
4	SP1161986	Kit, extension tube, cpl. RH		1
5	SP1161987	Kit, extension tube, cpl. LH		1
6	SP1161990	Footplate, cpl. RH		1
7	SP1161991	Footplate, cpl. LH		1
8	SP1163375	Footrest pad (Pair)		1
9	SP1579988	Support, Lever, cpl.		1
10	SP1579989	Support, Centr. Legrest man. LNX - SW 380+430mm		1
11	SP1579990	Support, Centr. Legrest man. LNX - SW 480+530mm		1
12	SP1579991	Support, Centr. Legrest man. LNX - SW 380+430mm, cpl.		1
13	SP1579992	Support, Centr. Legrest man. LNX - SW 480+530mm, cpl.		1

Central Legrest LNX electr. - Standard- / Easy Adapt-Seat


1543031X.iss

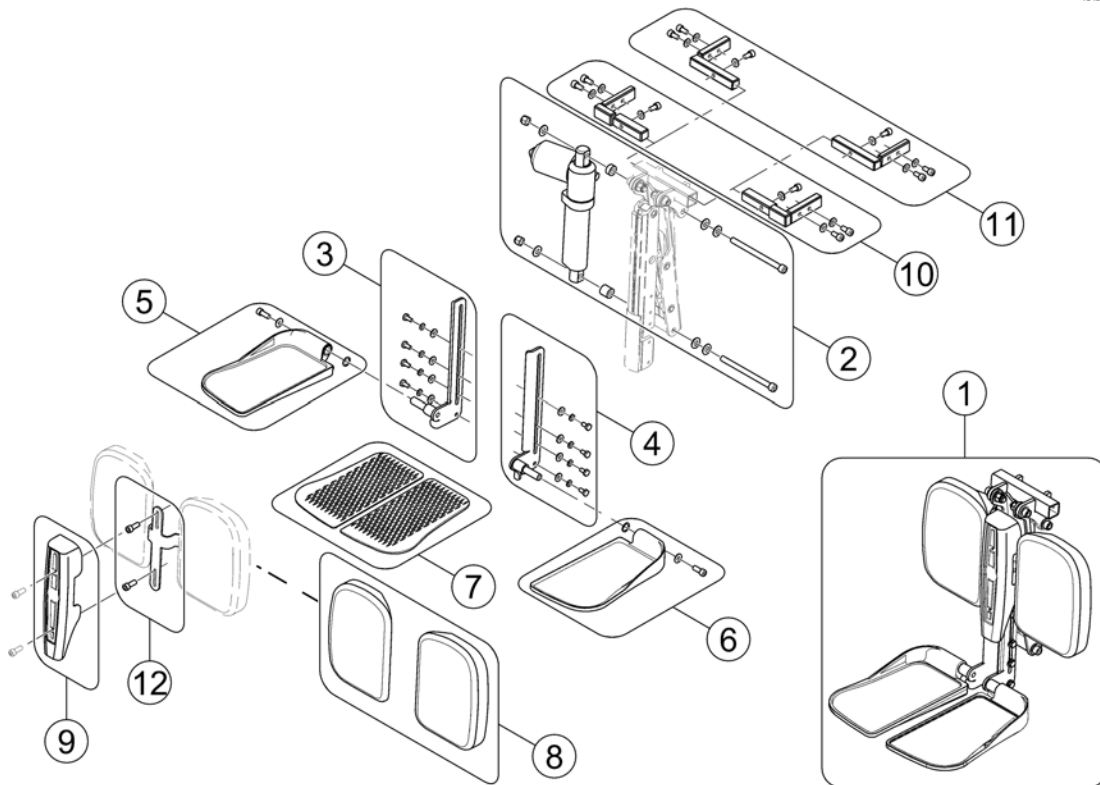

Pos.	Part number	Description	Valid from → until	Qty
-	SP1541677	Adaptercable 300mm <i>For chairs with lifter</i>		1
-	SP1543012	Adaptercable 1200mm <i>For chairs without lifter</i>		1
1	SP1537489	Centre Mounted Legrest LNX (Standard / Easy Adapt) complete		1
2	SP1543031	Actuator, complete		1
3	SP1543035	Footplate mount, RH		1
4	SP1543034	Footplate mount, LH		1
5	SP1543037	Footplate, RH		1
6	SP1543036	Footplate, LH		1
7	SP1163375	Footrest pad (Pair)		1
8	SP1543033	Calfplates (Pair)		1
9	SP1543032	Cover		1
10	SP1541397	Support center mounted legrest 430-530 mm		1
11	SP1541398	Support center mounted legrest 380-410 mm		1
12	SP1579985	Calf Plate Bracket		1


Central Legrest LNX electr. - Modulite Seat

1575133 iso

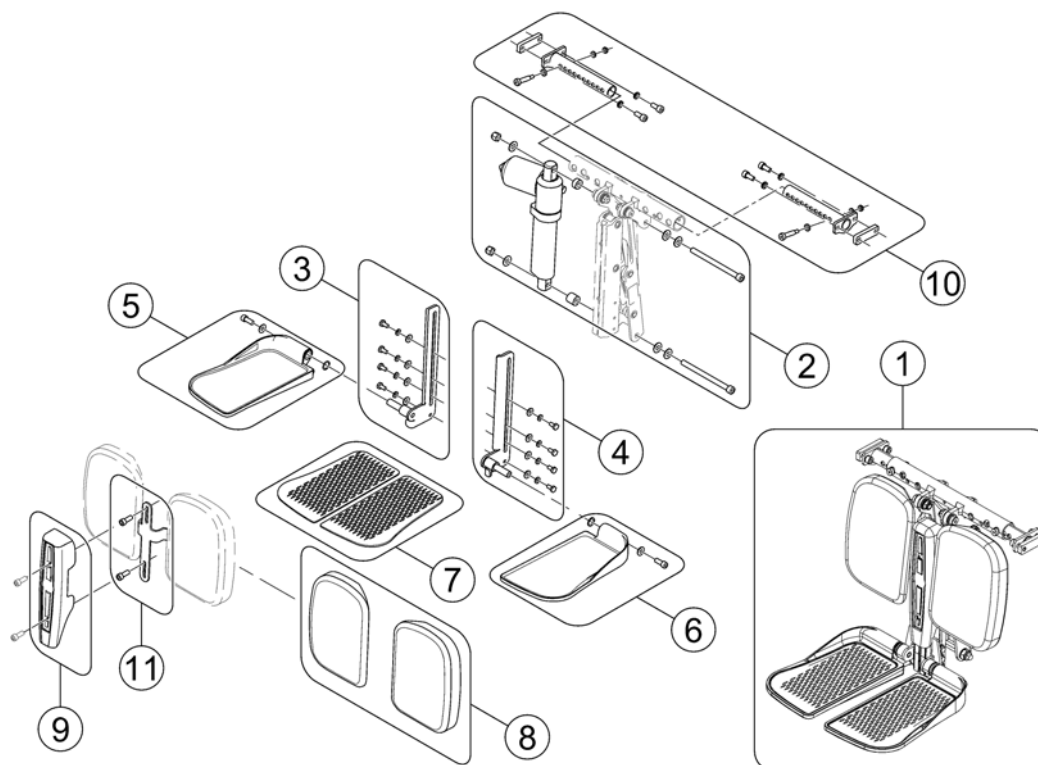

Pos.	Part number	Description	Valid from → until	Qty
-	SP1541677	Adaptercable 300mm <i>For chairs with lifter</i>		1
-	SP1543012	Adaptercable 1200mm <i>For chairs without lifter</i>		1
1	SP1571959	Centr. Legrest el. LNX for Modulite cpl. (incl. Support)		1
2	SP1543031	Actuator, complete		1
3	SP1543035	Footplate mount, RH		1
4	SP1543034	Footplate mount, LH		1
5	SP1543037	Footplate, RH		1
6	SP1543036	Footplate, LH		1
7	SP1163375	Footrest pad (Pair)		1
8	SP1543033	Calfplates (Pair)		1
9	SP1543032	Cover		1
10	SP1575144	Support, Centr. Legrest el. LNX - Modulite		1
11	SP1579985	Calf Plate Bracket		1


Centre-mounted Legrests

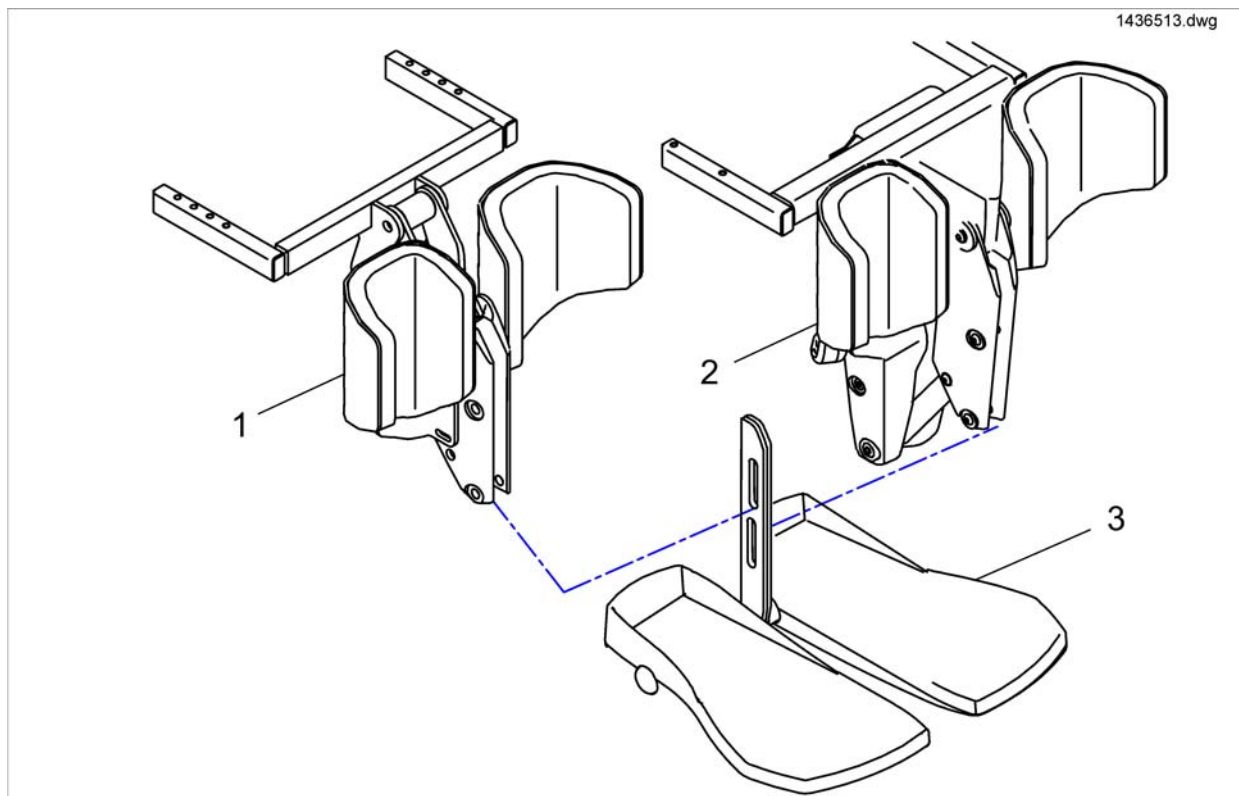

Pos.	Part number	Description	Valid from → until	Qty
1	SP1436512	Centre Hanger, manual, cpl.		1
2	SP1440413	CENTRAL HANGER 90°, ELECTRICAL, COMPLETE		1
3	SP1440292	Central Lower Legrest, large (44-50cm), cpl.		1
3	SP1440291	Central Lower Legrest, medium (38-44cm), cpl.		1
3	SP1440290	Central Lower Legrest, small (32-38cm), cpl.		1

Bracket Centr. Legrest LNX - Narrow Base


1541573X.iso

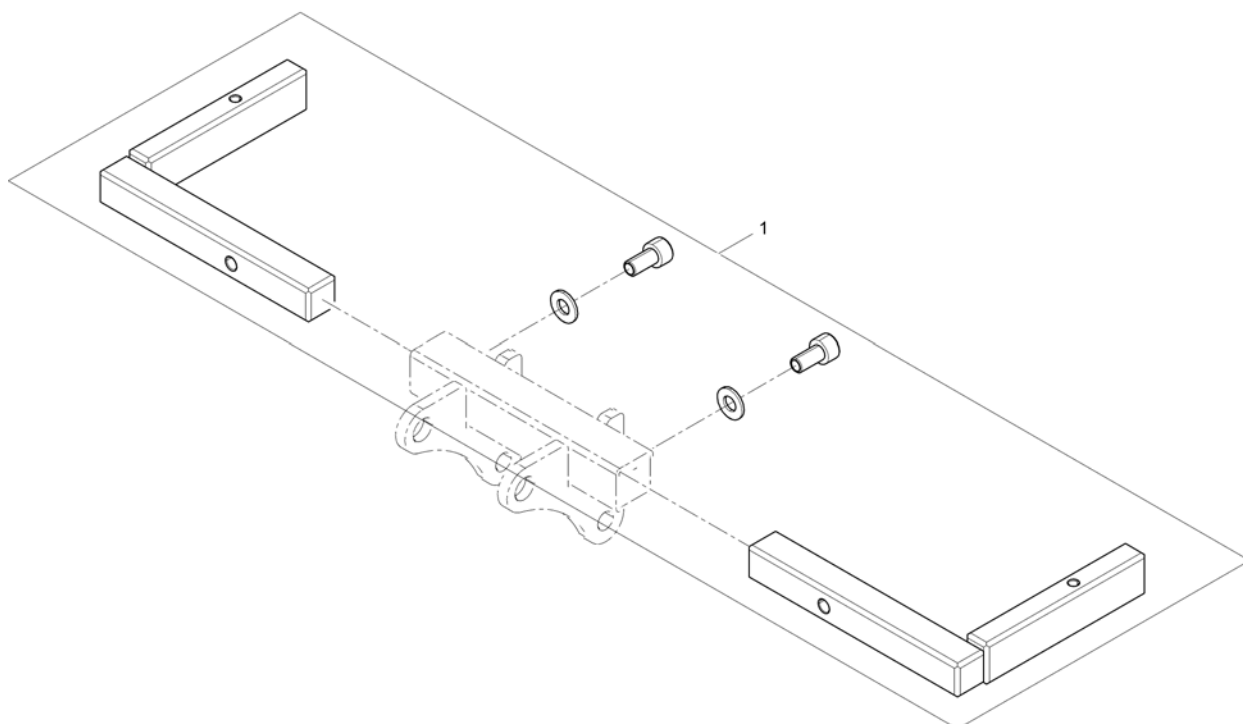

Pos.	Part number	Description	Valid from → until	Qty
1	SP1543042	Bracket central legrest LNX & manuell		1
1	SP1554509	Bracket central legrest LNX & manuell (Narrow base)		1

Footrests Vari, Flip-Up (01.08.10 =>)


1529644X.jpg

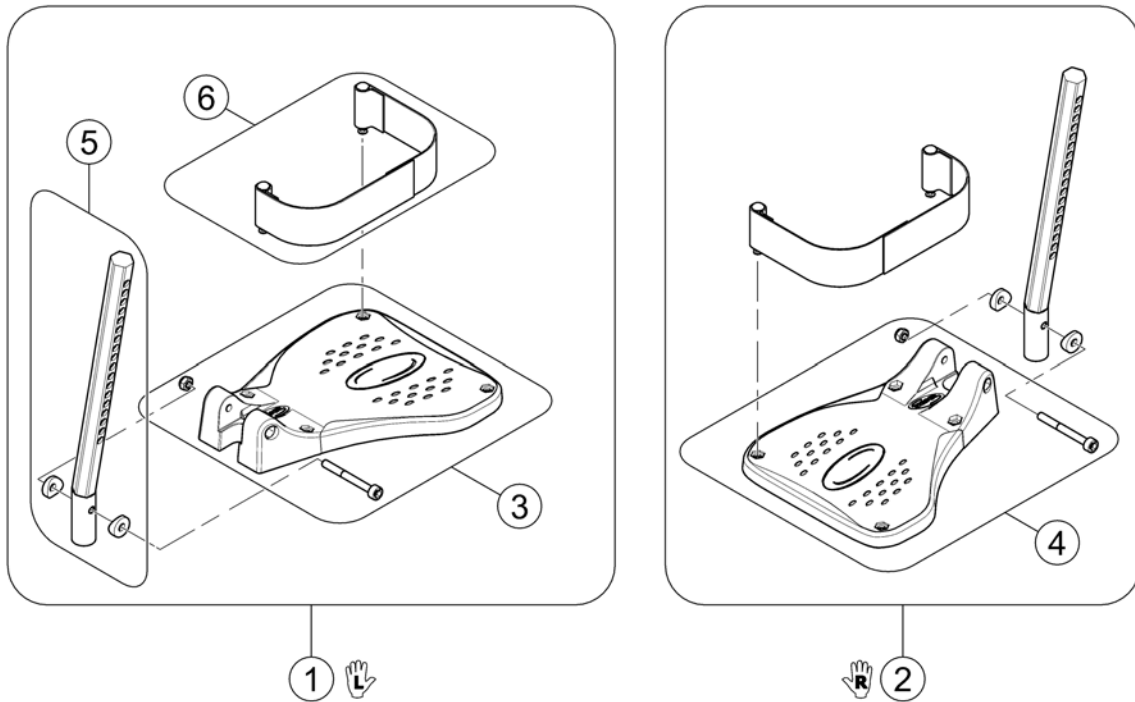

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541575	Footplate flip up, LH, cpl.		1
2	SP1541574	Footplate flip up, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1543045	Plug-in tube LH/RH		1
6	SP1442467	Heel Strap cpl.		1


Footrests Vari, Flip-Up & angle adjustable (01.08.10 =>)

152964EX.es

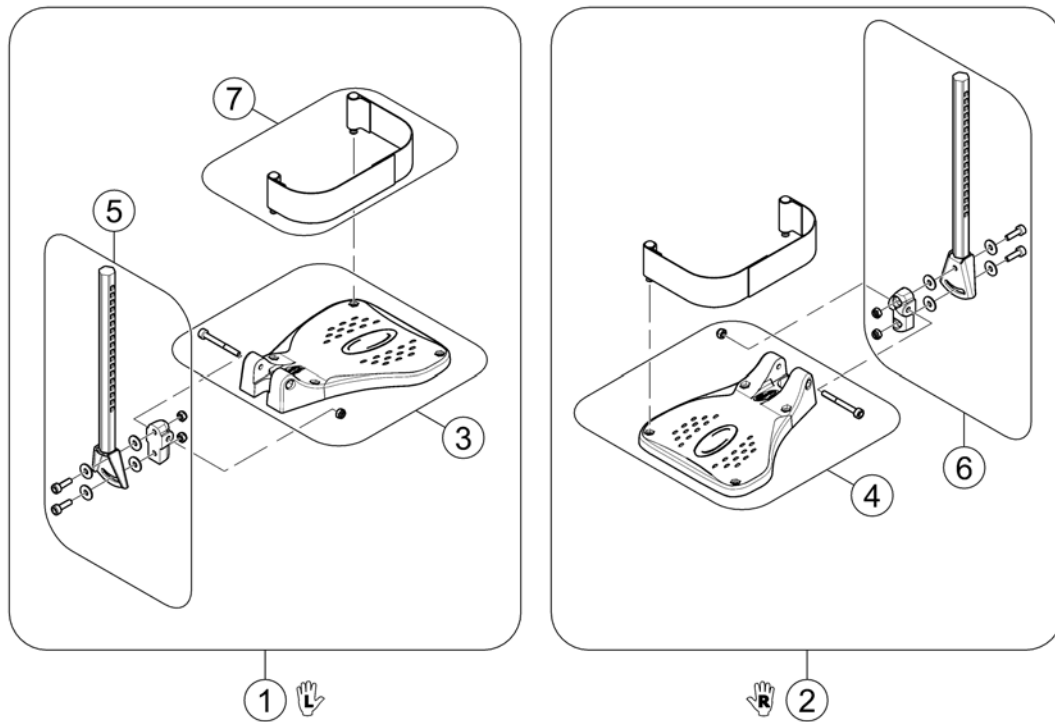


Pos.	Part number	Description	Valid from → until	Qty
1	SP1541580	Footplate flip up, angle adjustable, LH, cpl.		1
2	SP1541579	Footplate flip up, angle adjustable, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1423204	FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.		1
6	SP1423203	FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.		1
7	SP1442467	Heel Strap cpl.		1

Footplate

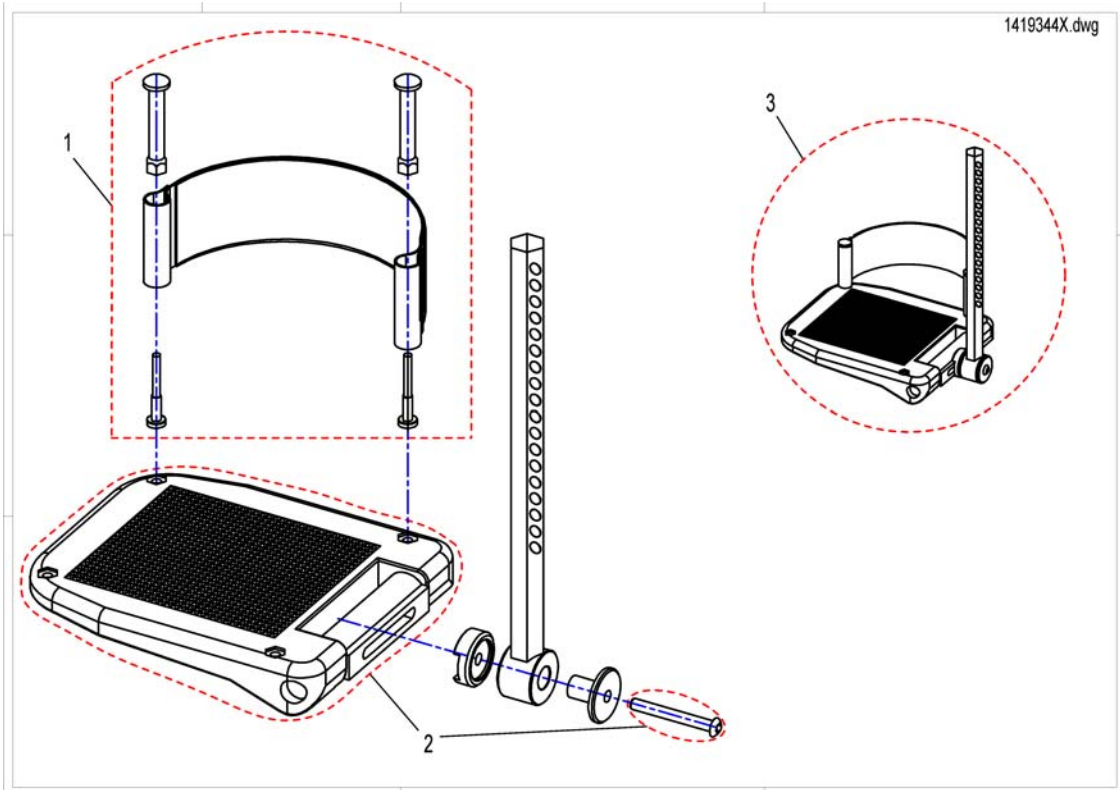


Pos.	Part number	Description	Valid from → until	Qty
1	SP5321579	Heel Strap		1
2	SP5106376	Footplate		1
3	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1
3	SP1419345	Footplate RH Anlge And Depth Adjustable, With Support Tube cpl.		1

Footplate with support tube fix RH/LH

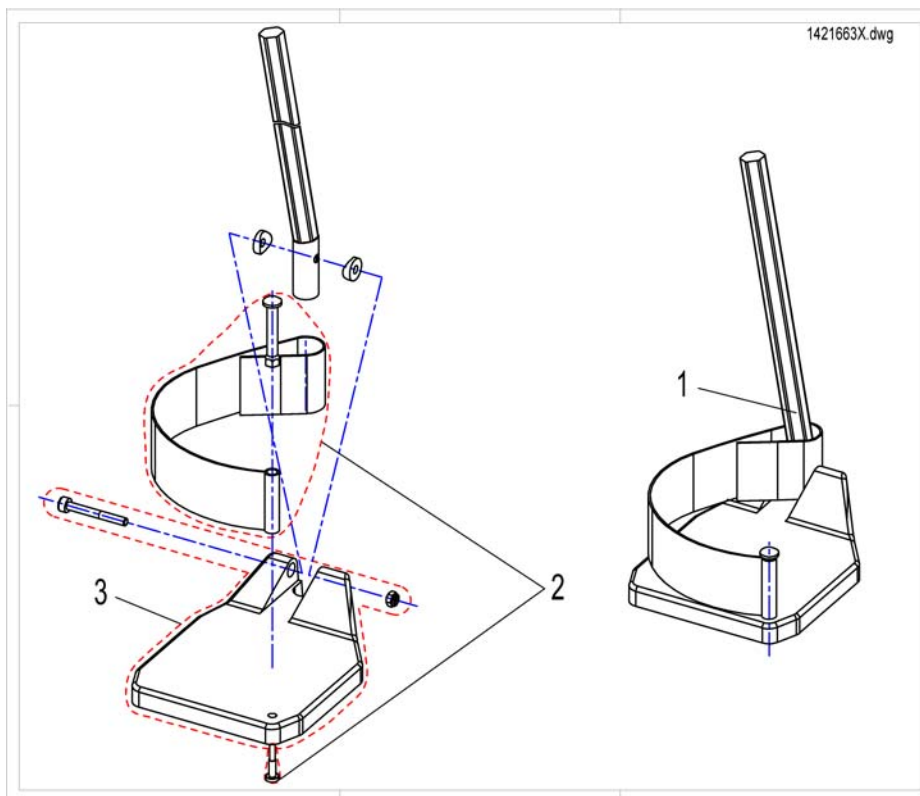

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421662	Footplate RH fix, with Support Tube cpl.		1
1	SP1421663	Footplate LH fix, with Support Tube cpl.		1
2	SPB273011	Heel Strap		1
3	SPZ173217	Footplate LH		1
3	SPZ173218	Footplate RH		1


Invacare® TDX-SP, TDX-SP nB, TDX Sprint | Footplates / Support tubes
Footplate with support tube angle adjust. RH/LH

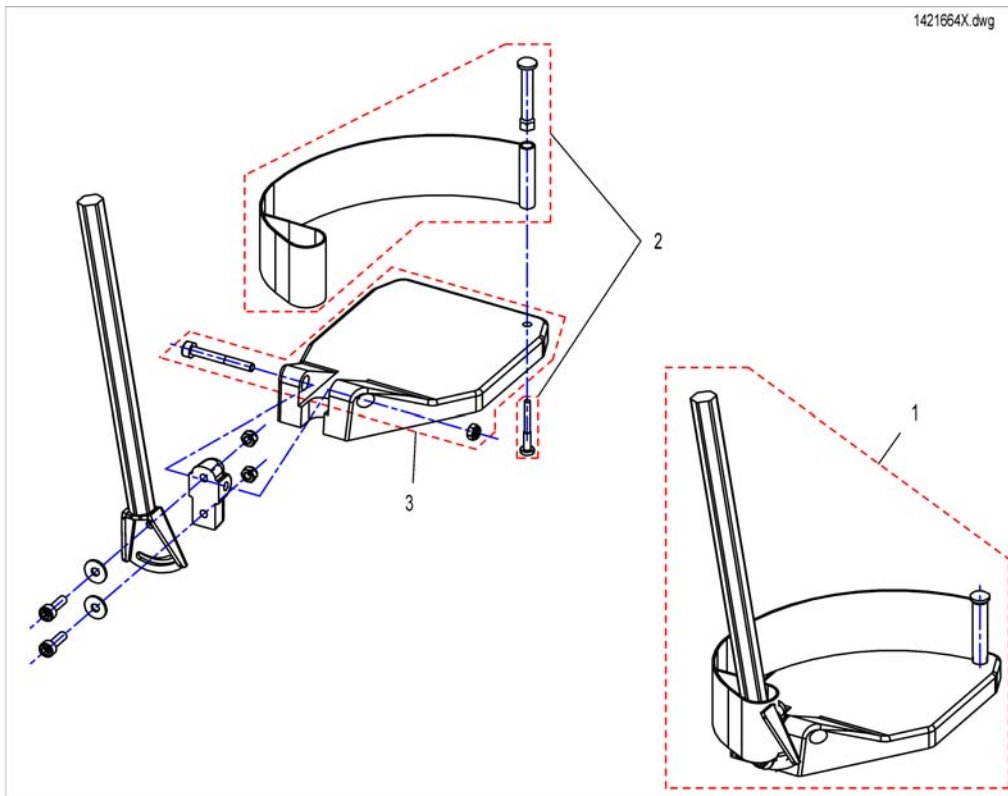

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421664	Footplate RH anlge adjustable, with Support Tube cpl.		1
1	SP1421665	Footplate LH anlge adjustable, with Support Tube cpl.		1
2	SPB273011	Heel Strap		1
3	SPZ173217	Footplate LH		1
3	SPZ173218	Footplate RH		1


Support Tube with Footplate for central Legrest

1440290X.dwg

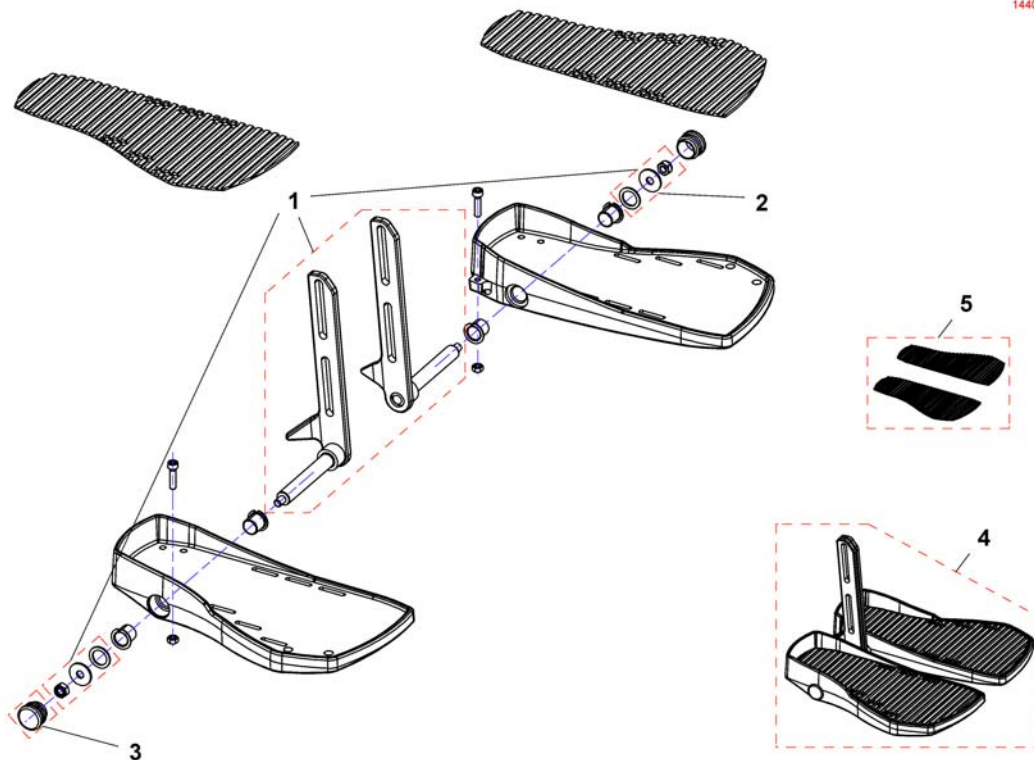


Pos.	Part number	Description	Valid from → until	Qty
1	SP1440442	Central Lower Legrest, small		1
1	SP1440443	Central Lower Legrest, medium		1
1	SP1440444	Central Lower Legrest, large		1
2	SPF00002	Fixation Kit		1
3	SP1440517	Darning		1
4	SP1440290	Central Lower Legrest, small (32-38cm), cpl.		1
4	SP1440291	Central Lower Legrest, medium (38-44cm), cpl.		1
4	SP1440292	Central Lower Legrest, large (44-50cm), cpl.		1
5	SP1546720	Footboardrubber		1

Footrest Lower Parts

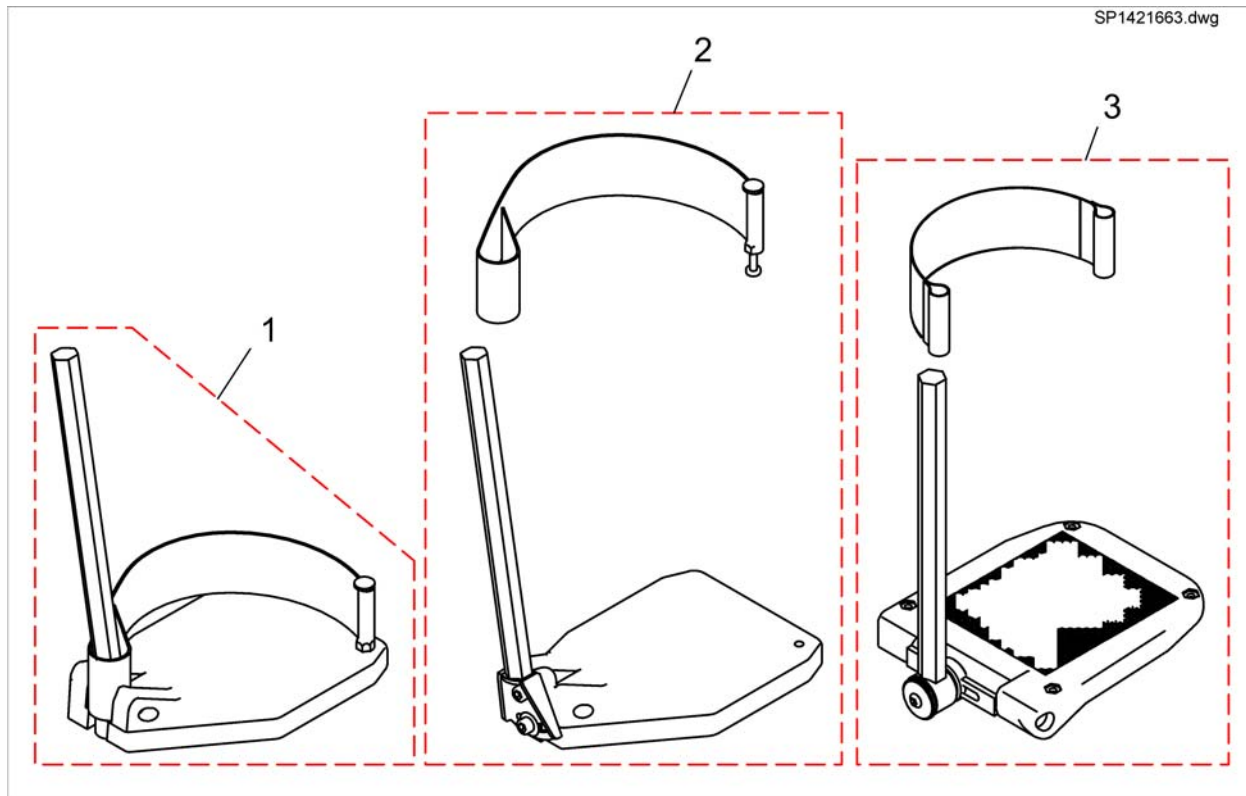

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421663	Footplate LH fix, with Support Tube cpl.		1
1	SP1421662	Footplate RH fix, with Support Tube cpl.		1
2	SP1421664	Footplate RH anlje adjustable, with Support Tube cpl.		1
2	SP1421665	Footplate LH anlje adjustable, with Support Tube cpl.		1
3	SP1419344	Footplate LH Anlje And Depth Adjustable, With Support Tube cpl.		1
3	SP1419345	Footplate RH Anlje And Depth Adjustable, With Support Tube cpl.		1
4	SP1446549	No description available.		1
5	SP1446548	No description available.		1

Armrests

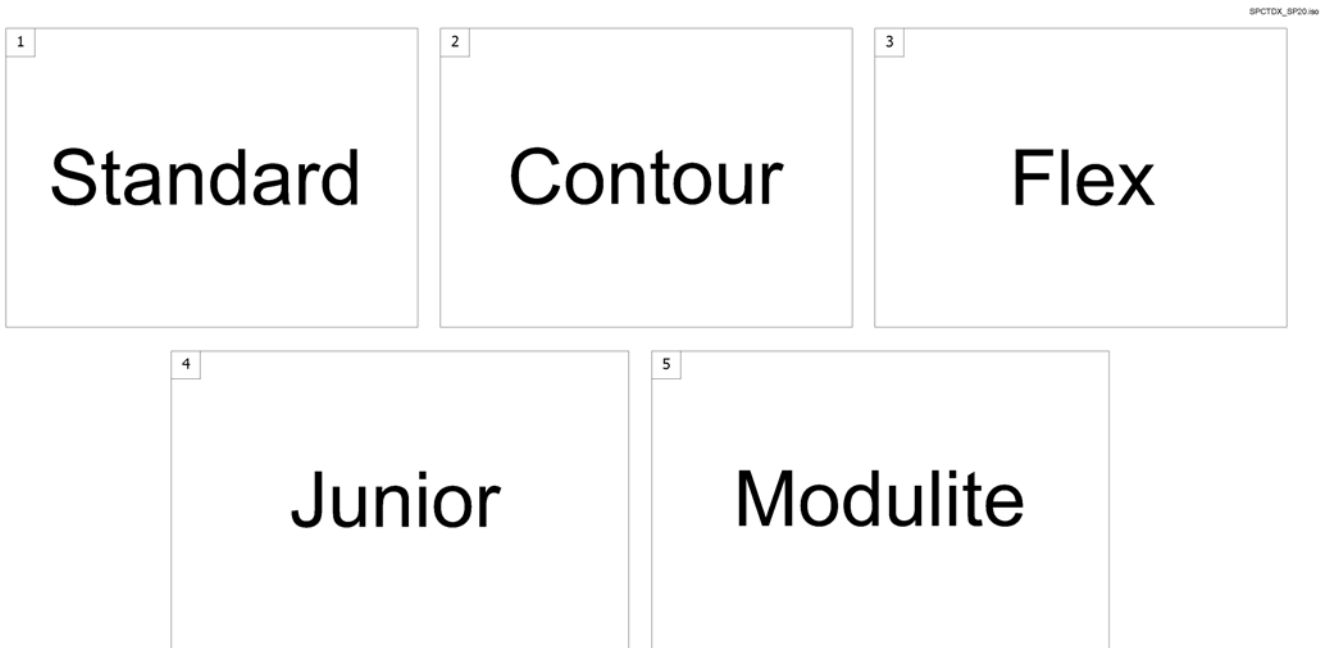


Pos.	Part number	Description	Valid from → until	Qty
1	-	Armrest for Standard Seat		1
2	-	Armrest for Contour Seat		1
3	-	Armrest for Flex Seat		1
4	-	Armrest for Junior Seat		1
5	-	Armrests for Modulite Seat		1

Armrest for Standard Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Armrest Standard [0635/0636]		1
-		Armrests cpl. [0635/0636]		1
-		T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==> [0606]		1

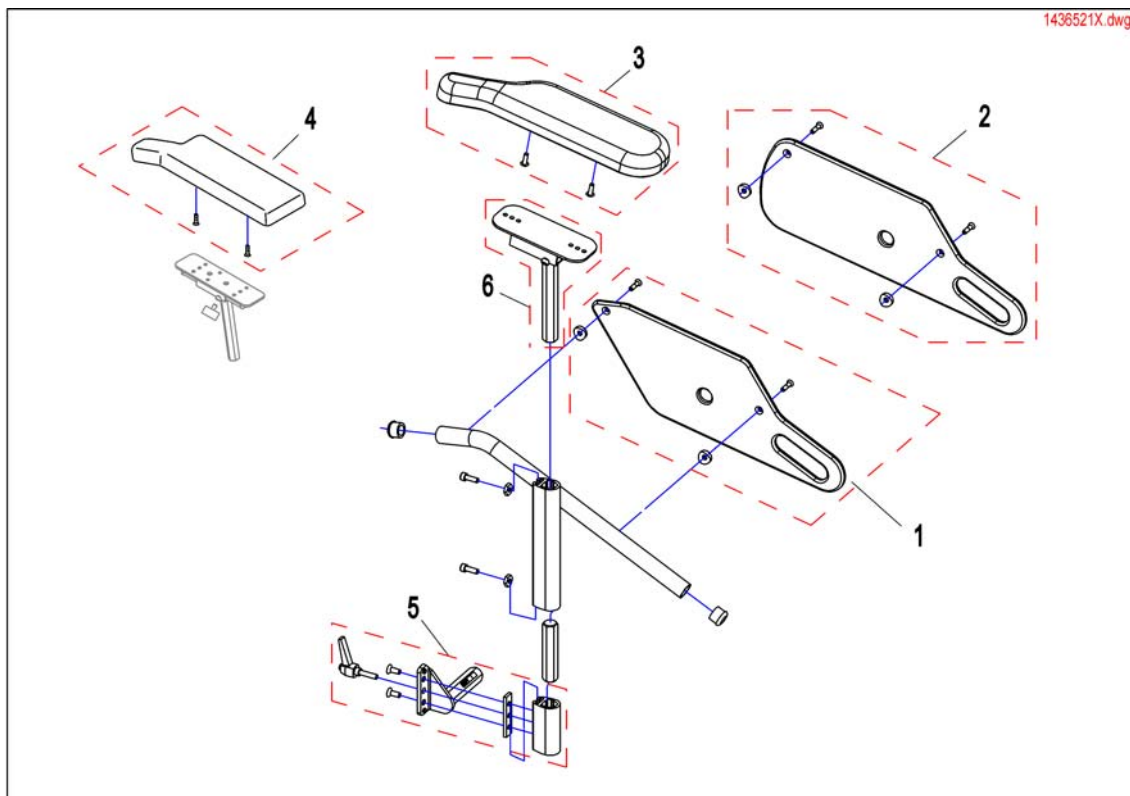

Armrest Standard



Pos.	Part number	Description	Valid from → until	Qty
1	SP1428078	Skirtguard, seat depth 410mm		1
2	SP1420471	Skirtguard, Seatdepth 460 & 510mm		1
3	SP1449043	Armpad, Standard (325mm), RH	2009-01-01 →	1
3	SP1443004	Armpad, Standard (325mm), LH	2009-01-01 →	1
3	SP1528845	Armpad, Extended (375mm), RH		1
3	SP1528846	Armpad, Extended (375mm), LH		1
4	SP1423221	ARMREST PAD RH CPL.	→ 2008-12-31	1
4	SP1423224	ARMREST PAD LH CPL.	→ 2008-12-31	1
5	SP1442550	Bracket for Armrest / Skirtguard RH		1
5	SP1442549	Bracket for Armrest / Skirtguard LH		1
6	SP1510857	Armpad holder LH		1
6	SP1510856	Armpad holder RH		1

Armrests cpl.


1421694X.dwg

Seat height 45cm/47cm Seatraiser - seatheight 47cm - 77cm					
Standard, Contour, Flex				01.01.09 ==>	
		Armrest height 340mm		Armrest height 380mm	
		RH	LH	RH	LH
> 39cm	> 41cm	1	2	3	4
39cm	> 41cm	5	6	7	8
> 39cm	<= 41cm	9	10	11	12
39cm	<= 41cm	13	14	15	16

child seat	
RH	LH
17	18

Pos.	Part number	Description	Valid from → until	Qty
1	SP1421692	Armrest, height max. 340mm, RH, Seatdepth 460mm & 510mm		1
2	SP1421694	Armrest, height max. 340mm, LH, Seatdepth 460mm & 510mm		1
3	SP1421693	Armrest, height max. 380mm, RH, Seatdepth 460mm & 510mm		1
4	SP1421695	Armrest, height max. 380mm, LH, Seatdepth 460mm & 510mm		1
5	SP1422542	Armrest, 340mm, SW 39cm, RH		1
6	SP1422543	Armrest, 340mm, SW 39cm, LH		1
7	SP1422548	Armrest, 380mm, SW 39cm, RH		1
8	SP1422549	Armrest, 380mm, SW 39cm, LH		1
9	SP1428096	Armrest, 340mm, SD 41cm, RH		1
10	SP1428097	Armrest, 340mm, SD 41cm, LH		1
11	SP1428098	Armrest, 380mm, SD 41cm, RH		1
12	SP1428099	Armrest, 380mm, SD 41cm, LH		1
13	SP1428100	Armrest, 340mm, SW 39cm, SD 41cm, RH		1
14	SP1428101	Armrest, 340mm, SW 39cm, SD 41cm, LH		1
15	SP1428102	Armrest, 380mm, SW 39cm, SD 41cm, RH		1
16	SP1428103	Armrest, 380mm, SW 39cm, SD 41cm, LH		1
17	5313557	Armrest Junior Seat RH		1
18	5313558	Armrest Junior Seat LH		1

T-Armrest, detachable, with Armpad, for Standard Seat 2008-08-01 ==>

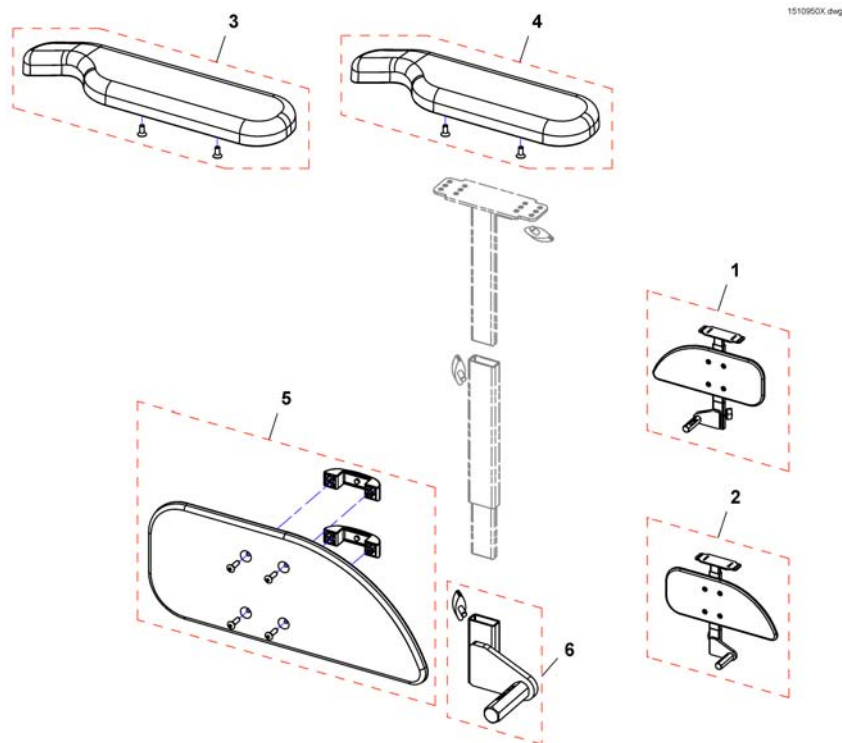


Pos.	Part number	Description	Valid from → until	Qty
1	SP1510935	Armrest, without armpad, RH		1
2	SP1510950	Armrest, without armpad, LH		1
3	SP1528846	Armpad, Extended (375mm), LH		1
3	SP1528845	Armpad, Extended (375mm), RH		1
4	SP1443004	Armpad, Standard (325mm), LH		1
4	SP1449043	Armpad, Standard (325mm), RH		1
5	SP1541505	Skirt guard, RH		1
5	SP1541506	Skirt guard, LH		1
6	SP1541510	Bracket for T-arm, RH		1
6	SP1541511	Bracket for T-arm, LH		1

Armrest for Contour Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Armrest Standard		1
-		Armrests cpl.		1
-		Mounting bracket Flip-up armrests		1
-		Following armrests		1
-		Sliding Armrest		1
-		Skirtguard for following armrest		1


Mounting bracket Flip-up armrests

1548568.dwg

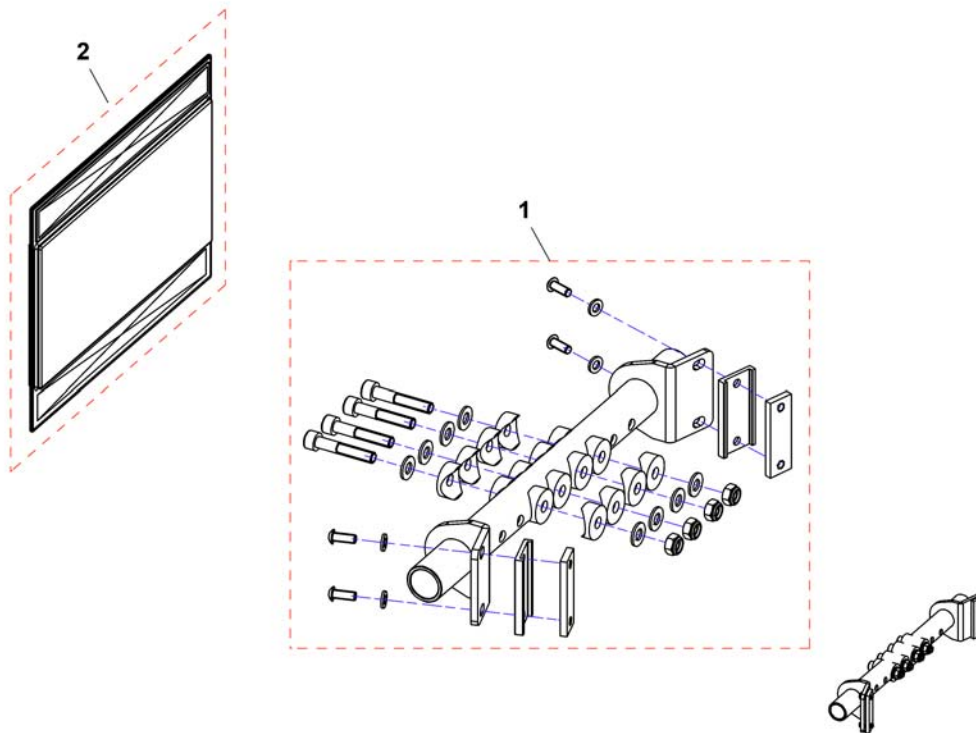


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548568	Bracket for flip-up armrest, SW 39		1
1	SP1548569	Bracket for flip-up armrest, SW 44		1
1	SP1548631	Bracket for flip-up armrest, SW 49		1
2	SP1550410	Protection cover		1

Following armrests

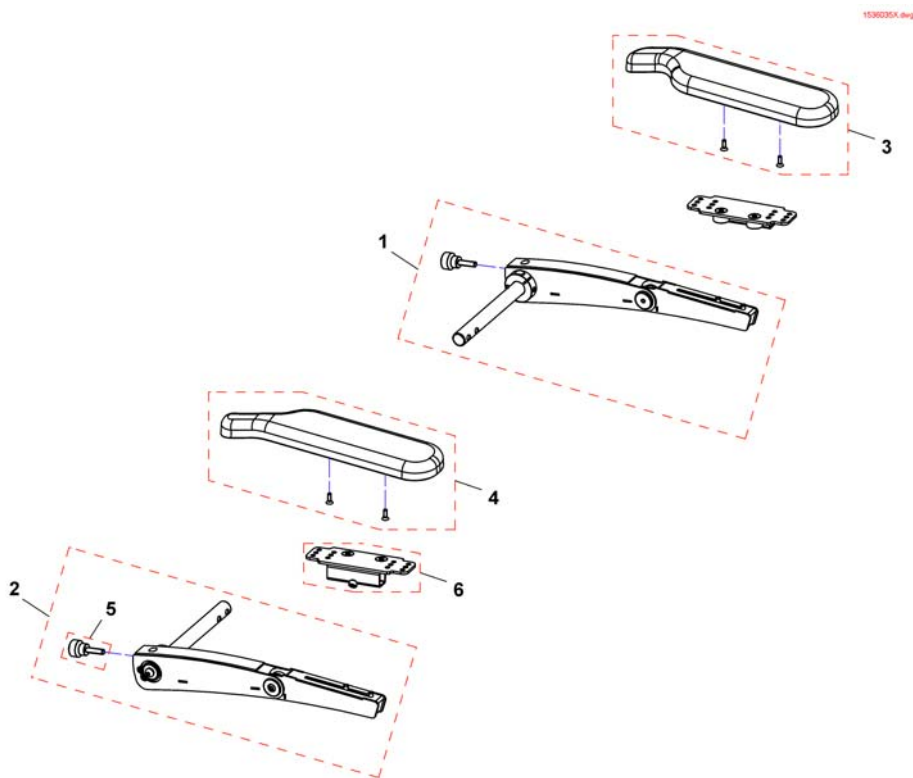

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536036	Following armrest, short, LH, cpl.		1
1	SP1536100	Following armrest, long, LH, cpl.		1
2	SP1536035	Following armrest, short, RH, cpl.		1
2	SP1536099	Following armrest, long, RH, cpl.		1
3	SP1443004	Armpad, Standard (325mm), LH		1
3	SP1528846	Armpad, Extended (375mm), LH		1
4	SP1449043	Armpad, Standard (325mm), RH		1
4	SP1528845	Armpad, Extended (375mm), RH		1
5	SP1535504	Knurled screw M8x30		1
6	SP1535244	Armpad holder		1


Sliding Armrest

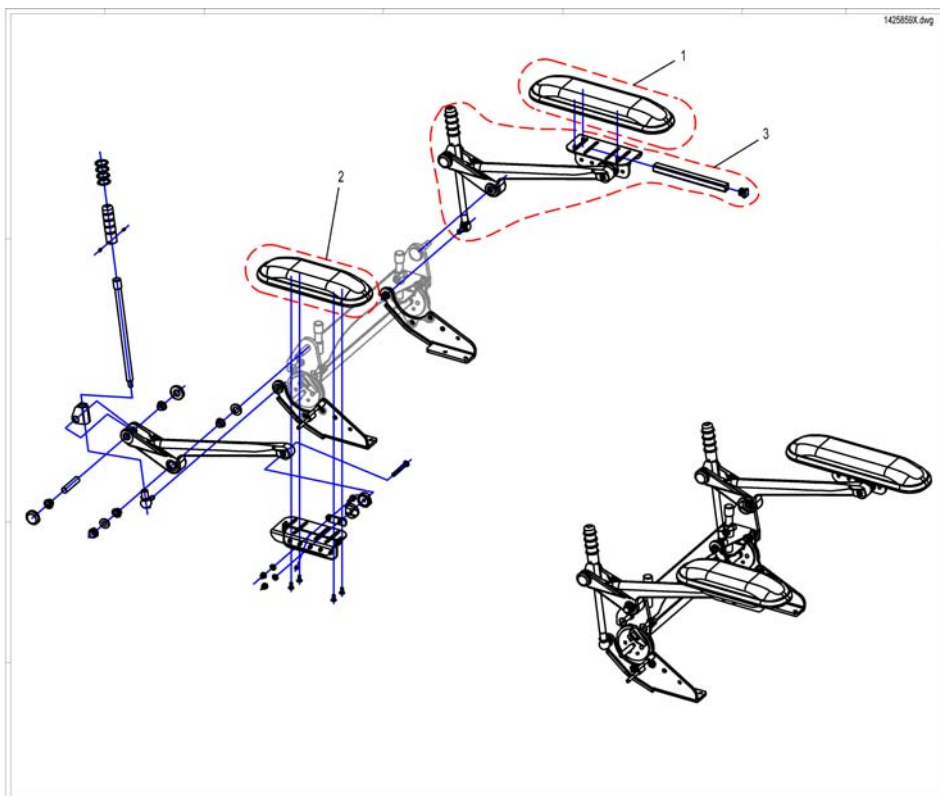

Pos.	Part number	Description	Valid from → until	Qty
1	SP1513484	Armrest long		1
2	SP1513485	Armrest short		1
3	SP1425844	ALU-Armrest		1

Skirtguard for following armrest

1425860X.dwg

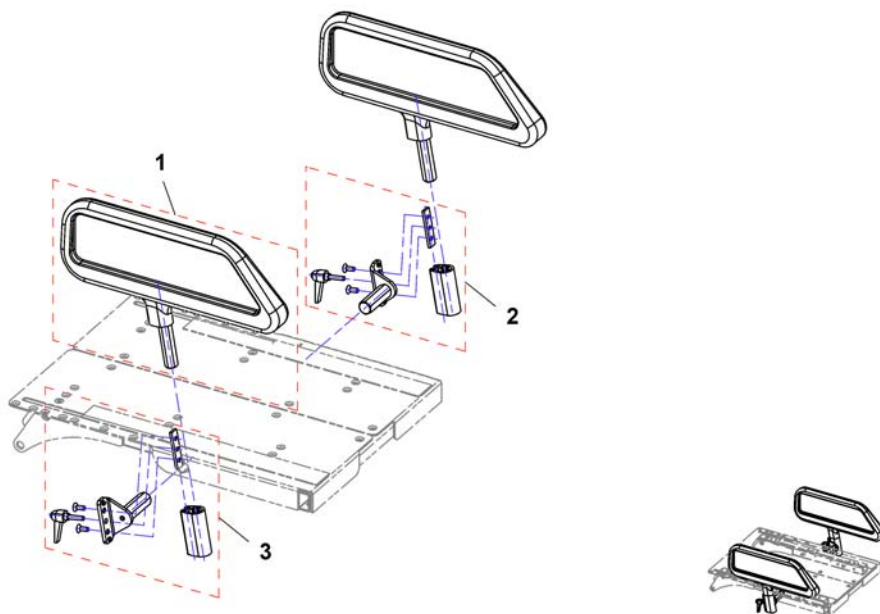

Pos.	Part number	Description	Valid from → until	Qty
1	SP1426303	Skirtguard		1
2	SP1442549	Bracket for Armrest / Skirtguard LH		1
3	SP1442550	Bracket for Armrest / Skirtguard RH		1

Armrest for Flex Seat


Pos.	Part number	Description	Valid from → until	Qty
-		Armrest Standard		1
-		Armrests cpl.		1
-		Mounting bracket Flip-up armrests		1
-		Following armrests		1
-		Sliding Armrest		1
-		Skirtguard for following armrest		1

Armrest for Junior Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Armrests cpl.		1

Armrests for Modulte Seat


1571738.000

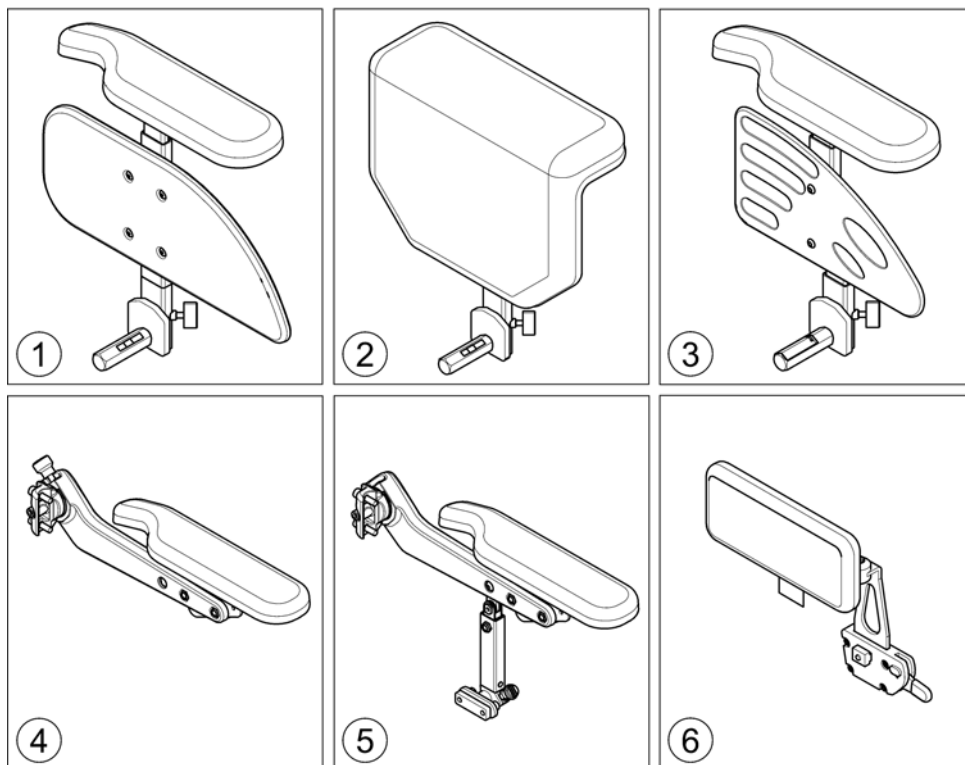

Pos.	Part number	Description	Valid from → until	Qty
1	-	T-Armrest [2191+2195/2196/2197]		1
2	-	Padded Armrest [2198]		1
3	-	Inrigged Armrest [2183+2195/2196/2197/2199]		0
4	-	Flip-Up Armrest [2193+2195/2196/2197/2199]		0
5	-	Following Armrest [2192/2194+2195/2196/2197/2199]		0
6	-	Hip Support [2184/2185/2186/2187/2188/2189]		0

T-Armrest


1572570.iso

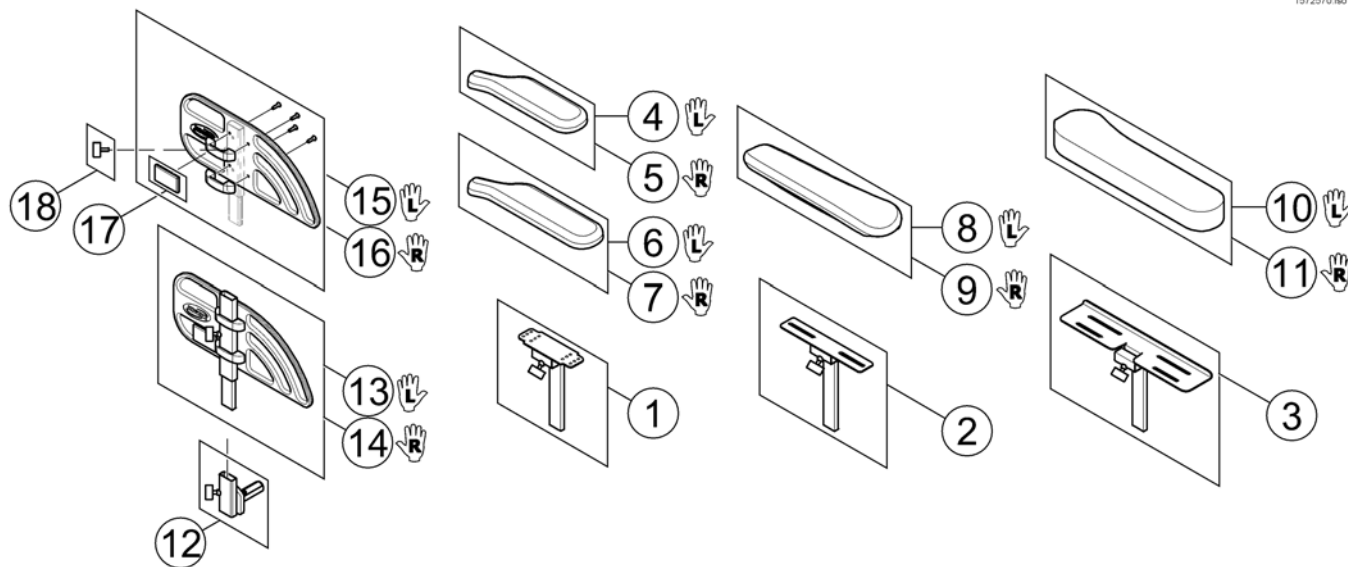

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1567749	Siderest Bracket - LH/RH		1
13	SP1588374	Siderest LH		1
14	SP1588375	Siderest RH		1
15	SP1572940	Skirt Guard LH		1
16	SP1572948	Skirt Guard RH		1
17	SPP31298	Reflector, yellow		1
18	SPL10024	Wing Screw M6x16		1

Padded Armrest


1573214.00

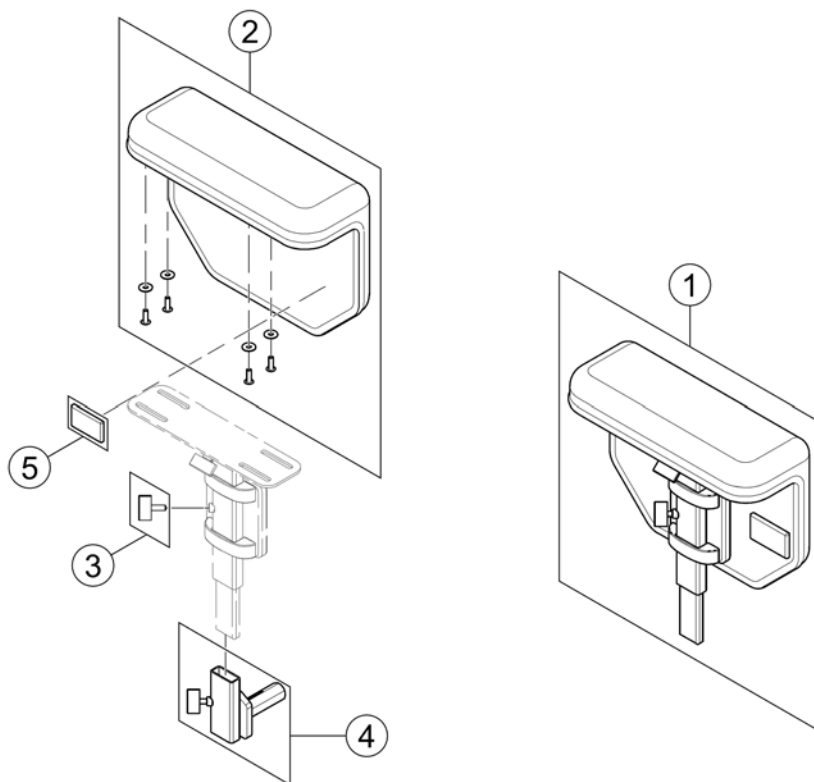

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571987	Padded Armrest RH/LH, cpl.		1
2	SP1571962	Armpad, padded - RH/LH with integrated cloth guard		1
3	SPL10024	Wing Screw M6x16		1
4	SP1567749	Siderest Bracket - LH/RH		1
5	SPP31298	Reflector, yellow		1


Inrigged Armrest

1585180 iso

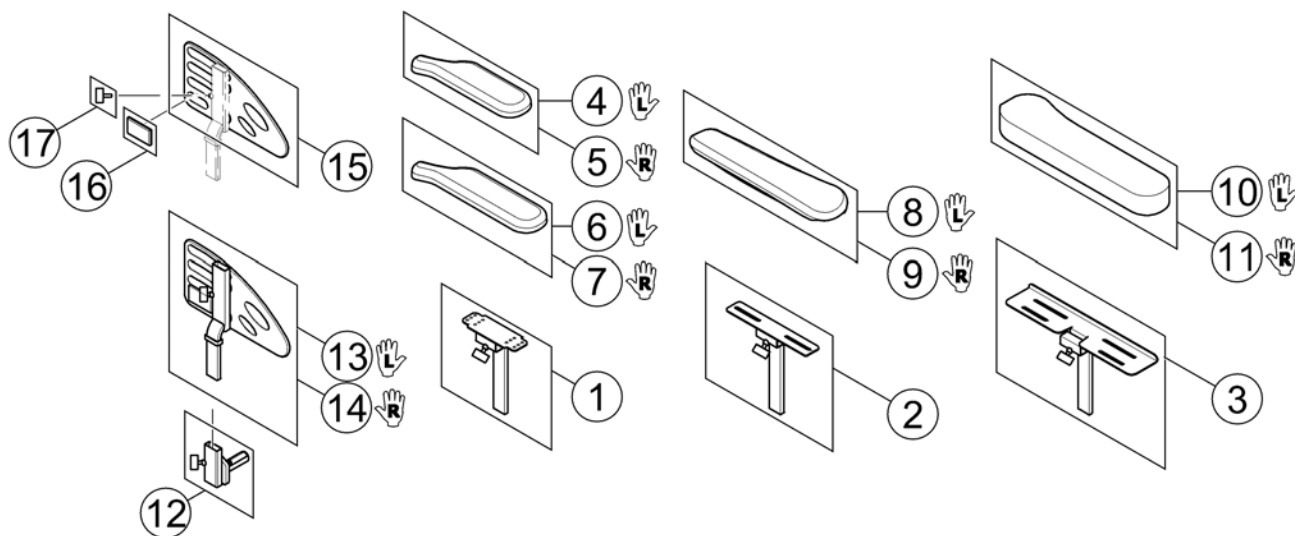

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567750	Armrest Bracket for Standard & Extended Armpad		1
2	SP1570319	Armrest Bracket for Ergonomic Armpad		1
3	SP1585088	Armrest Bracket for Soft Armpad		1
4	SP1443004	Armpad, Standard (325mm), LH		1
5	SP1449043	Armpad, Standard (325mm), RH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1528845	Armpad, Extended (375mm), RH		1
8	SP1571589	Armpad, Ergonomic, LH		1
9	SP1571588	Armpad, Ergonomic, RH		1
10	SP1585068	Armpad, Soft, LH		1
11	SP1585069	Armpad, Soft, RH		1
12	SP1567749	Siderest Bracket - LH/RH		1
13	SP1585037	Siderest inrigged LH		1
14	SP1585038	Siderest inrigged RH		1
15	SP1578119	Skirt Guard LH/RH		1
16	SPP31298	Reflector, yellow		1
17	SPL10024	Wing Screw M6x16		1


Flip-Up Armrest

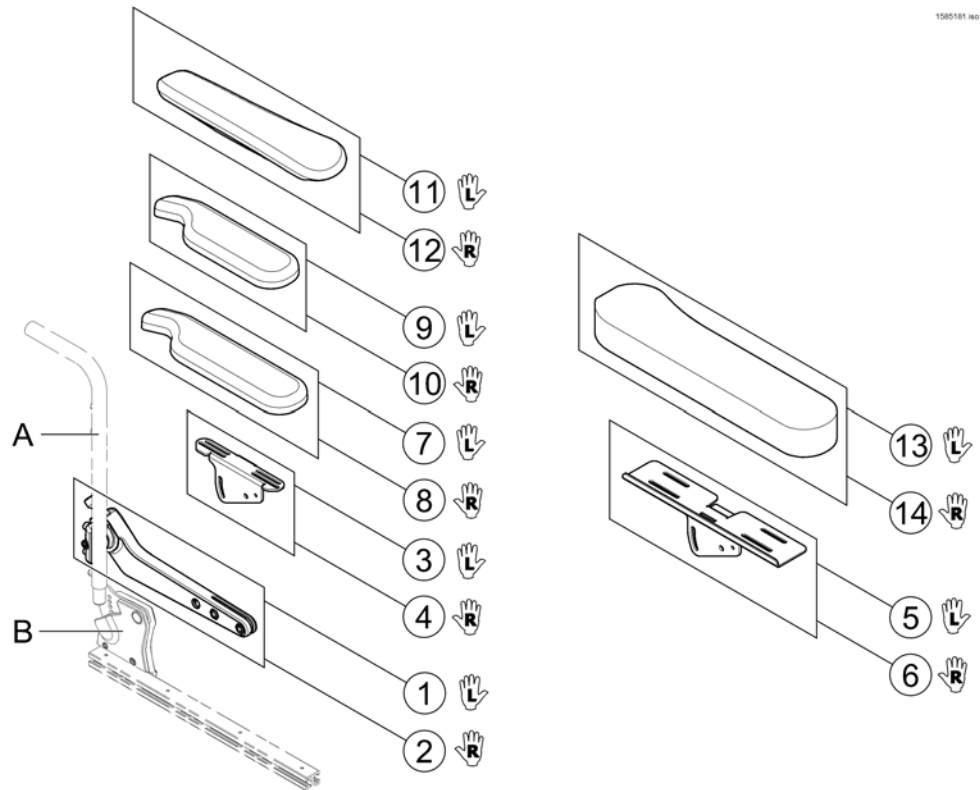

1580181 00

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1582668	Flip-Up Armrest, LH	2015-01-19 →	1
2	SP1582644	Flip-Up Armrest, RH	2015-01-19 →	1
3	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
4	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
5	SP1585414	Ampad Support (Soft), LH		1
6	SP1585415	Ampad Support (Soft), RH		1
7	SP1528846	Ampad, Extended (375mm), LH		1
8	SP1528845	Ampad, Extended (375mm), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1571589	Ampad, Ergonomic, LH		1
12	SP1571588	Ampad, Ergonomic, RH		1
13	SP1585068	Ampad, Soft, LH		1
14	SP1585069	Ampad, Soft, RH		1


Following Armrest

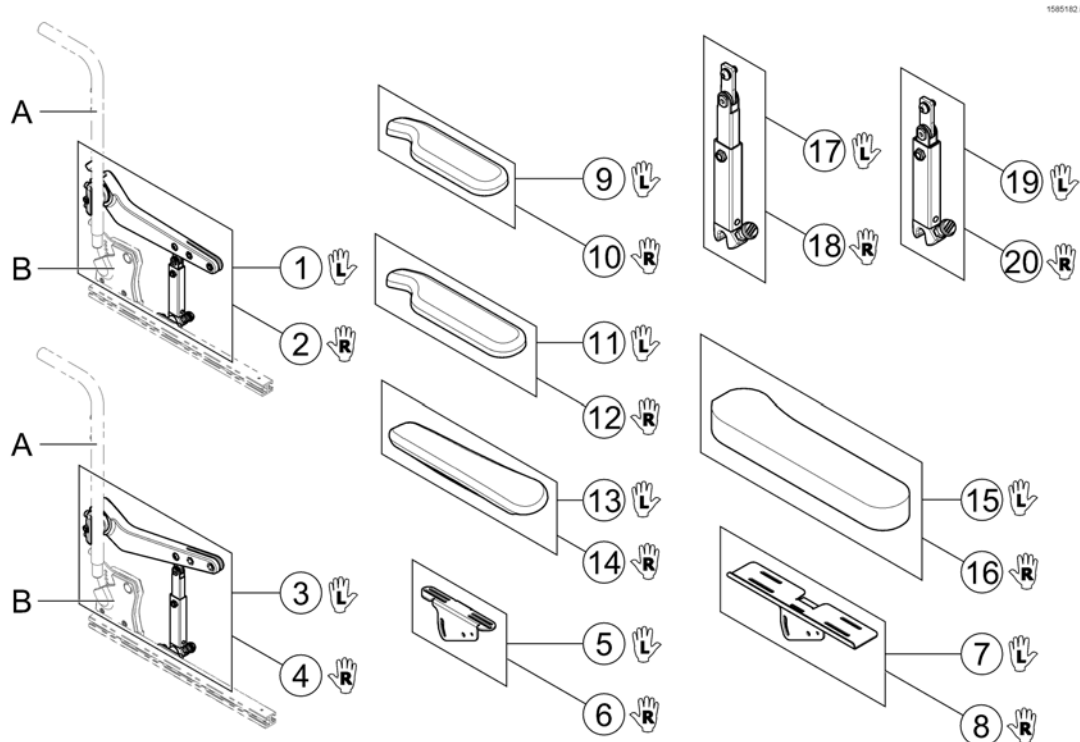

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1585263	Armrest LH cpl. - Short Support	2015-01-19 →	1
2	SP1585264	Armrest RH cpl. - Short Support	2015-01-19 →	1
3	SP1585267	Armrest LH cpl. - Long Support	2015-01-19 →	1
4	SP1585268	Armrest RH cpl. - Long Support	2015-01-19 →	1
5	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
6	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
7	SP1585414	Ampad Support (Soft), LH		1
8	SP1585415	Ampad Support (Soft), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1528846	Ampad, Extended (375mm), LH		1
12	SP1528845	Ampad, Extended (375mm), RH		1
13	SP1571589	Ampad, Ergonomic, LH		1
14	SP1571588	Ampad, Ergonomic, RH		1
15	SP1585068	Ampad, Soft, LH		1
16	SP1585069	Ampad, Soft, RH		1
17	SP1583156	Amrest Support, long, LH		1
18	SP1583155	Amrest Support, long, RH		1
19	SP1583154	Amrest Support, short, LH		1
20	SP1582639	Amrest Support, short, RH		1

Hip Support


1585183 Rev

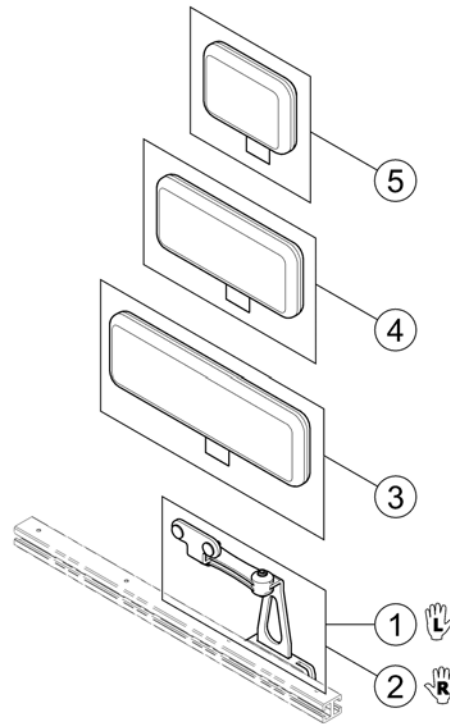

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584411	Hip Support, Quick Release bracket, LH		1
2	SP1584410	Hip Support, Quick Release bracket, RH		1
3	SP1584414	Hip Support Pad, large (330x100 mm)		1
4	SP1584413	Hip Support Pad, medium (230x100 mm)		1
5	SP1584412	Hip Support Pad, small (140x100 mm)		1


Castors Front/Rear

1142342.00

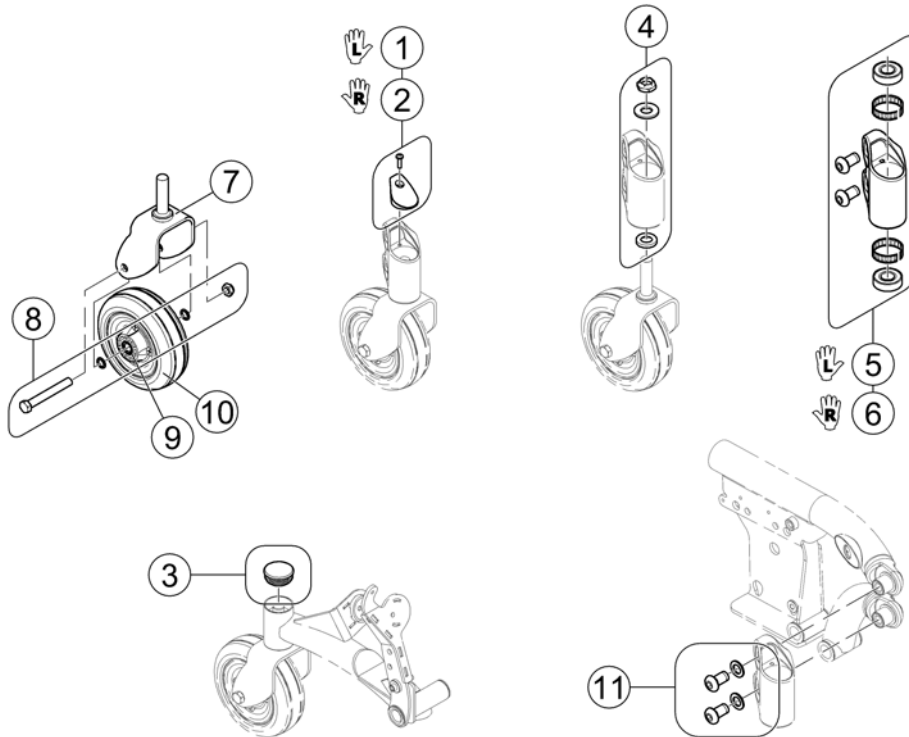

Pos.	Part number	Description	Valid from → until	Qty
-	SP1537420	Spacer, Stem (5/8 x 1-1/8 x 3/16)"		1
1	SP1523141	Kit, Cap with Screw, Front Head Tube -Left		1
2	SP1523140	Kit, Cap with Screw, Front Head Tube -Right		1
3	SP1523093	Kit, Cap with Screw, Rear Head Tube -Right/Left		1
4	SP1522999	Kit, Fork Mounting Hardware		1
5	SP1523139	Kit, Front Head Tube Assembly-Left		1
6	SP1523094	Kit, Front Head Tube Assembly-Right		1
7	SP1523016	Fork and Stem For TRRO/TRBKTS Option		1
8	SP1537431	Kit, Castor Mounting Hardware		1
9	SP1522935	Spacer, Bearing (7/16 x 5/8 x 1-3/4")		1
10	SP1175881	6x2" Wheel with bearings, silver hub		1
10	SP1511369	6x2" Wheel with bearings, black hub		1
11	SP1523176	Kit, Front Head Tube Mounting Hardware		1


Drive Wheels

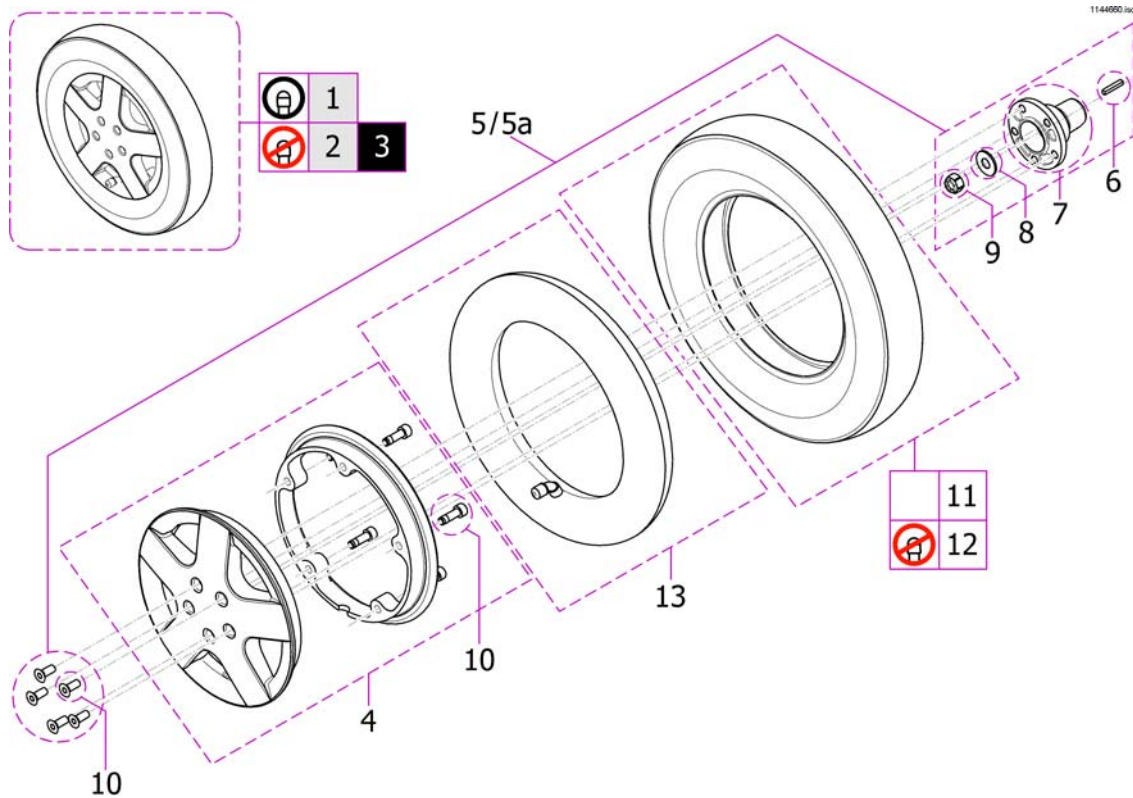

Pos.	Part number	Description	Valid from → until	Qty
1	SP1532414	Wheel 14 x 3.00" pneumatic, silver, complete		1
2	SP1532416	Wheel 14 x 3.00" puncture proof, silver, complete		1
3	SP1523161	Wheel 14 x 3.00" puncture proof, black, complete		1
4	SP1523013	Rim-Kit for 14 x 3" Wheel, silver, complete		1
5	SP1523157	Drive Wheel 14 x 3" Mounting Kit		1
5a	SP1168890	Drive Wheel 14 x 3" Mounting Kit (2-Pole Motor!)		1
6	SP1522934	Key (3/16 x 1-1/4")		1
7	SP1523092	Wheel Hub		1
8	SP1522939	Washer (1/2 x 1-9/32 x 1/8")		1
9	1025737	Axle Locknut 1/2"-20		1
10	SP1522975	Bolt (5/16-18 x 7/8")		1
11	SP1522940	Tire 14 x 3.00"		1
12	SP1141342	Tire 14 x 3.00" puncture proof		1
13	SP1039651	Tube 14"		1

Motor & Gearbox ==> 06.09.11


1144057 Rev

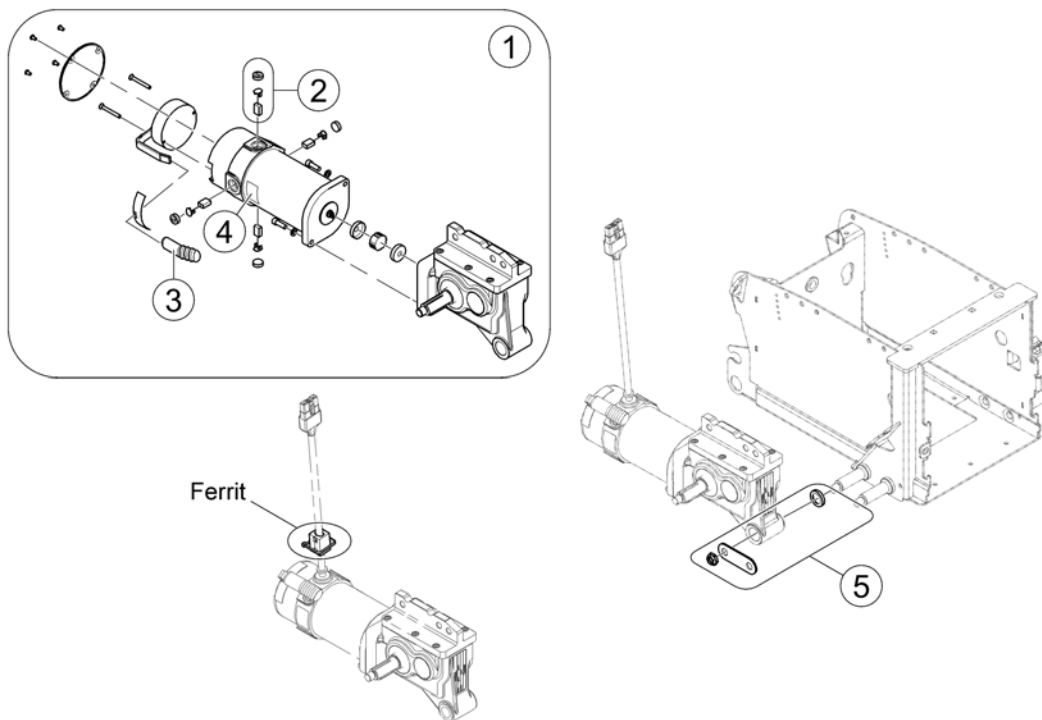

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523164	Motor & Gearbox Assembly 8km/h, LH		1
1	SP1523163	Motor & Gearbox Assembly 8km/h, RH		1
1	SP1144657	Motor & Gearbox Assembly 9.5km/h, LH		1
1	SP1144656	Motor & Gearbox Assembly 9.5km/h, RH		1
2	SP1522967	Kit, brushes with caps, 4-pole (Set of 4)		4
3	SP1522976	Cap, Declutching Lever		1
4	1104797	Disengaging information label		1
5	SP1523168	Kit, Lower Pivot Link / 4-Pole Motor Mounting Hardware		1

Overview Propulsion SSD

1163090X.iso

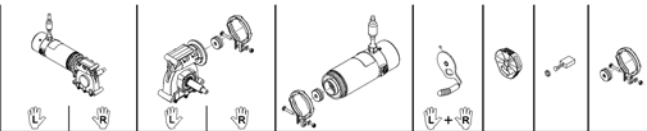

Bora/ SpectraXTR	6km/h	c	ACS/ Shark	1	2	33	34	37	---	51	45
			VR2	3	4	33	34	43	---	51	45
			R-Net	5	6	33	34	41	---	51	45
	10km/h	d	ACS/ Shark	7	8	31	32	36	---	51	45
			VR2	9	10	31	32	40	---	51	45
			R-Net	11	12	31	32	42	---	51	45
BoraHD/ Spectra XTR2HD	6km/h		ACS/ Shark	54	55	33	34	39	---	51	46
	10km/h		ACS/ Shark	56	57	31	32	38	---	51	46
	6km/h		R-Net	58	59	33	34	62	---	51	46
	10km/h		R-Net	60	61	31	32	63	---	51	46
FDX	6km/h	e		13	14	33	34	37	53	---	45
	6km/h	f	ACS/ Shark	54	55	33	34	39	53	---	46
	10km/h			15	16	31	32	38	53	---	46
Kite	6km/h		ACS/ Shark	17	18	33	34	37	---	51	45
	10km/h		ACS/ Shark	19	20	31	32	36	---	51	45
Kite HD	6km/h		ACS/ Shark	21	22	33	34	39	---	51	46
	10km/h		ACS/ Shark	23	24	31	32	38	---	51	46
Storm4/ Storm4 X-plore	6km/h		ACS/ Shark	47	48	34	33	39	---	51	46
	10km/h		ACS/ Shark	27	28	32	31	38	---	51	46
TDX-SP	6km/h	a		25	26	34	33	39	53	---	46
	8km/h	a	ACS/ Shark	29	30	34	33	38	53	---	46
	10km/h	b		49	50	32	31	38	53	---	46
TDX-SP Narrow	6km/h		ACS/ Shark	25	26	34	33	39	53	---	46
	10km/h		ACS/ Shark	49	50	32	31	38	53	---	46

a) used from 28.09.11 / b) used from 07.09.11 / c) used from 18.04.11 / d) used from 01.08.11 / e) used till 31.05.13 / f) used from 01.06.13 (S/N 13P06000078)



Pos.	Part number	Description	Valid from → until	Qty
-	N23004	Feather key DIN6885-A-6x6x40		1
1	SP1524195	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
2	SP1524194	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
3	SP1541408	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
4	SP1541410	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
5	SP1541409	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
6	SP1541411	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
7	SP1524197	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
8	SP1524196	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
9	SP1541404	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
10	SP1541406	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
11	SP1541405	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
12	SP1541407	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
13	SP1168983	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
14	SP1168984	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
15	SP1168989	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
16	SP1168988	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
17	SP1568147	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1

Overview Propulsion SSD

Pos.	Part number	Description	Valid from → until	Qty
18	SP1568148	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
19	SP1568149	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
20	SP1568150	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
21	SP1568151	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
22	SP1568152	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
23	SP1568153	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
24	SP1568208	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
25	SP1536022	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
26	SP1536021	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
27	SP1541366	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
28	SP1541365	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =></i>		1
29	SP1170307	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
30	SP1170306	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
31	SP1163413	Gearbox, LH, i = 18:1		1
32	SP1163412	Gearbox, RH, i = 18:1		1
33	SP1163415	Gearbox, LH, i = 23:1		1
34	SP1163414	Gearbox, RH, i = 23:1		1
35	SP1542953	Clamp band with clutch		1
36	SP1163099	Motor, 2-pole, 250W, fast, 600mm cable, ACS		1
37	SP1164046	Motor, 2-pole, 250W, slow, 600mm cable, ACS		1
38	SP1165863	Motor, 4-pole, 340W, fast, 760mm cable, ACS		1
39	SP1165865	Motor, 4-pole, 340W, slow, 760mm cable, ACS		1
40	SP1544840	Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
41	SP1544796	Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
42	SP1544841	Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
43	SP1544795	Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
45	SP1540358	Kit, brushes with caps, 2-pole (Set of 2)		1
46	SP1522967	Kit, brushes with caps, 4-pole (Set of 4)		1
47	SP1583821	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable <i>S/N 061122812868 =></i>		1
48	SP1583822	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable <i>S/N 061122812868 =></i>		1
49	SP1534060	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
50	SP1534059	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
51	SP1579595	Declutching knob SSD RH/LH (1 pc) <i>Single part. Consider to buy another one for the other side if required. - Replaces SP1168668, SP1168669 & SP1169005</i>		1
53	SP1169004	Declutching lever, complete set (2 pcs.)		1
54	SP1552782	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1
55	SP1552783	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, ACS		1

Overview Propulsion SSD


Pos.	Part number	Description	Valid from → until	Qty
56	SP1552872	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
57	SP1552873	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, ACS		1
58	SP1552832	Propulsion cpl. LH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
59	SP1552833	Propulsion cpl. RH, Gearbox i = 23:1, Motor 4-pole, 340W, slow, 760mm cable, R-Net		1
60	SP1552836	Propulsion cpl. LH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
61	SP1552837	Propulsion cpl. RH, Gearbox i = 18:1, Motor 4-pole, 340W, fast, 760mm cable, R-Net		1
62	SP1562498	Motor, 4-Pol, 340W, slow, 760mm cabel, R-Net		1
63	SP1562499	Motor, 4-Pol, 340W, fast, 760mm cabel, R-Net		1

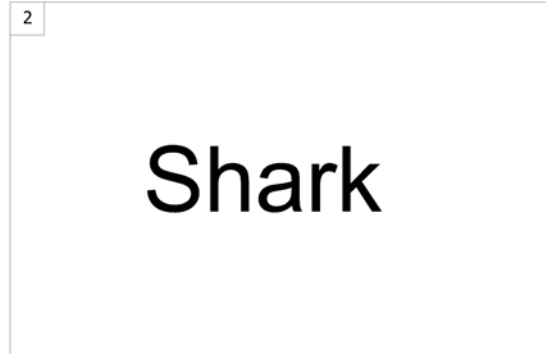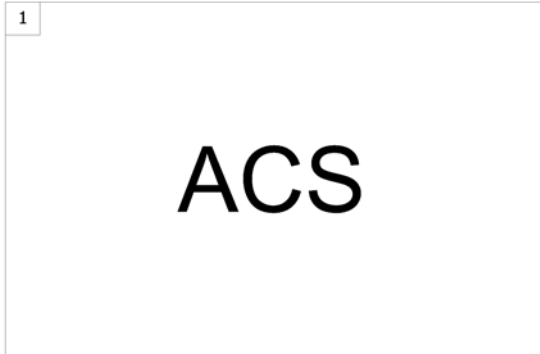

Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic		1
2	-	Shark Electronic		1

ACS Electronic

1500784.sp

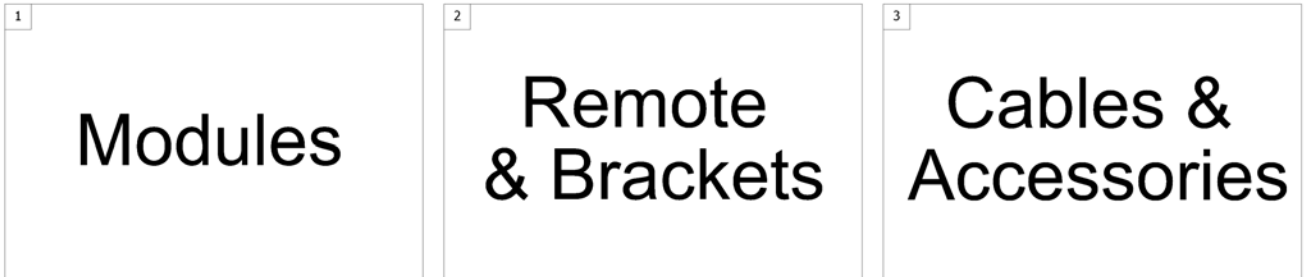


Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic Modules		1
2	-	ACS Remote & Brackets		1
3	-	ACS Cables & Accessories		1

ACS Electronic Modules

Pos.	Part number	Description	Valid from → until	Qty
-		Powermodule ACS2		1
-		ACT2/4 for ACS2		1
-		Bracket ACT narrow base		1
-		Lightboard ACS2		1
-		G-Trac-sensor		1
-		Holder for G-Trac		1

Powermodule ACS2


1529381.dwg

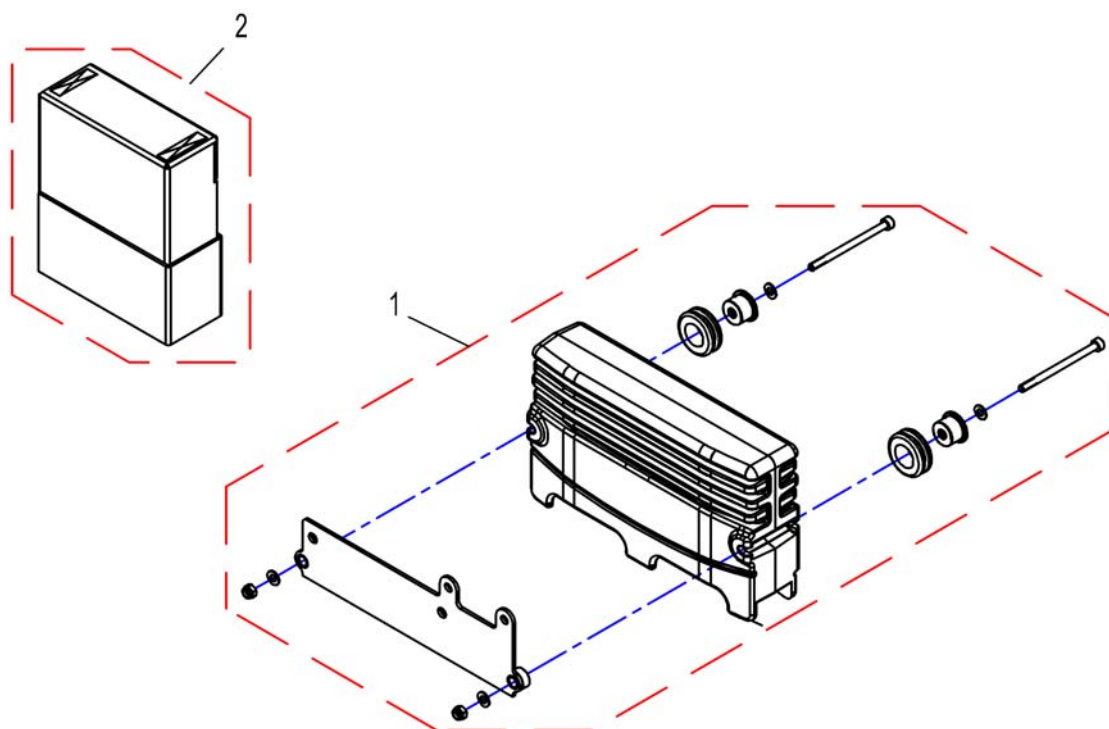

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529381	Powermodule ACS2 DX2-PMA90L (Light)		1
1	SP1541534	Powermodule ACS2 DX2-PMA90LG (G-Trac)		1
2	SP1532518	Mudguard		1

ACT2/4 for ACS2


153314/X.DWG

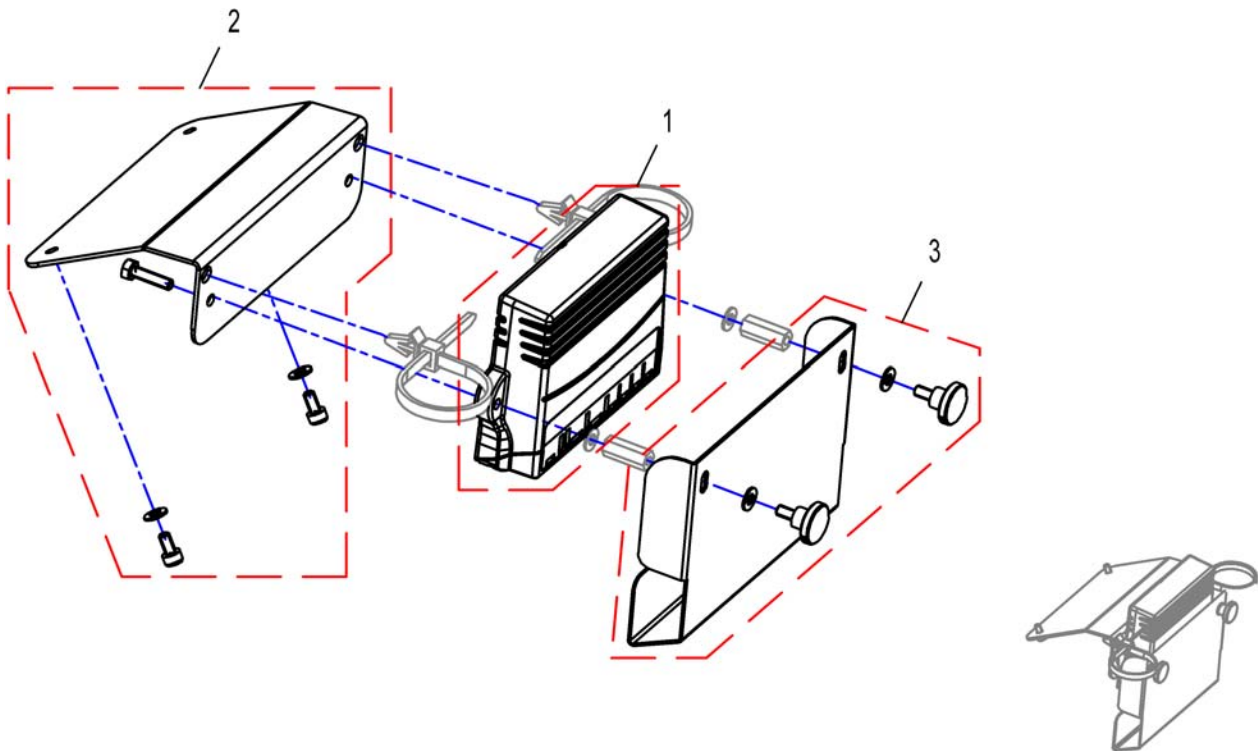

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529382	ACT4 for ACS2, cpl.		1
1	SP1534955	ACT2 for ACS2, cpl.		1
2	SP1536213	Mudguard ACT		1
3	SP1536273	ACT-Holder		1
-	SP1536274	Fastening-Kit (such as all bolts, cable ties)		1

Bracket ACT narrow base


151183X.dwg

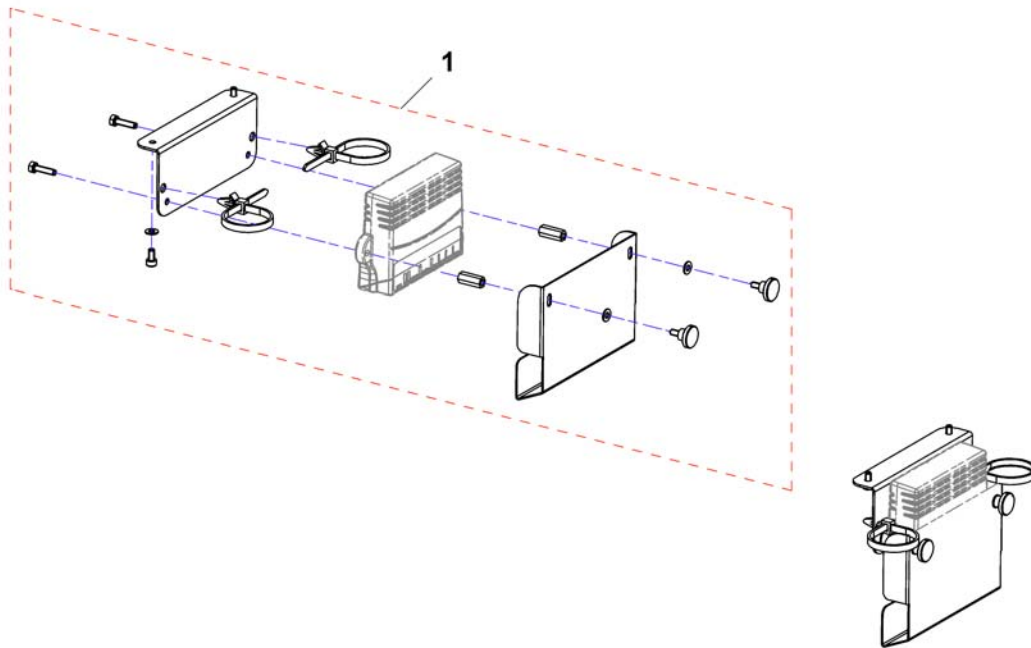

Pos.	Part number	Description	Valid from → until	Qty
1	SP1554510	Bracket ACT narrow base		1

Lightboard ACS2


1529383.dwg

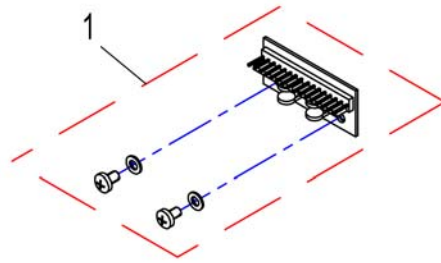

Pos.	Part number	Description	Valid from → until	Qty
1	SP1529383	Lightboard ACS2		1

G-Trac-sensor


1537848X.dwg

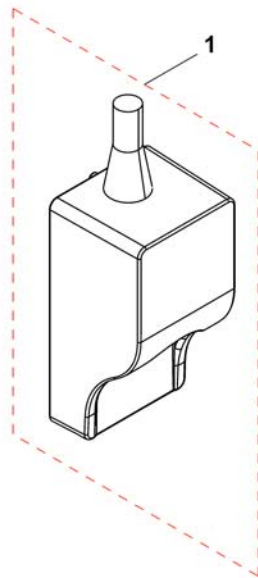

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541535	G-Trac		1


Holder for G-Trac

1541535X.dwg

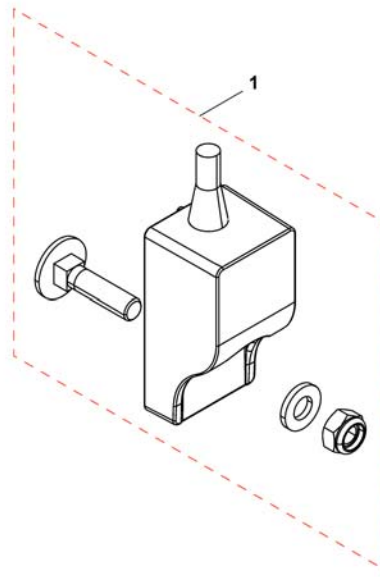

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541752	Holder for G-Trac		1


ACS Remote & Brackets

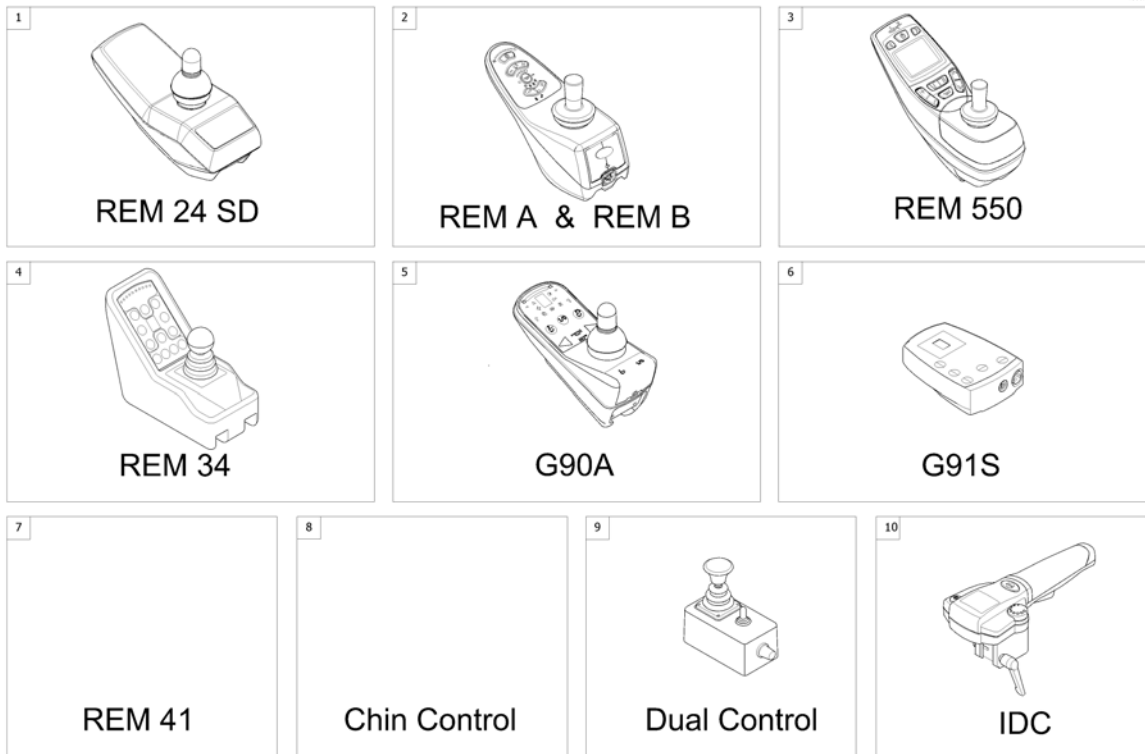

Pos.	Part number	Description	Valid from → until	Qty
1	-	REM 24 SD with brackets <i>[1601/1602]</i>		1
2	-	REM A & B with brackets <i>[1638/1639/1643/1644]</i>		1
3	-	REM 550 with brackets <i>[1617/1618]</i>		1
4	-	REM 34 with brackets <i>[1611]</i>		1
5	-	G90A with brackets <i>[1614/1616]</i>		1
6	-	G91S with brackets <i>[1615]</i>		1
7	-	Centre control REM 41 <i>[1607/1608]</i>		1
8	-	Small Chin Control with brackets <i>[1609]</i>		1
9	-	Remote Dual Control with Brackets <i>[1291/1292]</i>		1
10	-	Intuitive Dual Control with brackets <i>[1645]</i>		1

REM 24 SD with brackets


1562656 iso

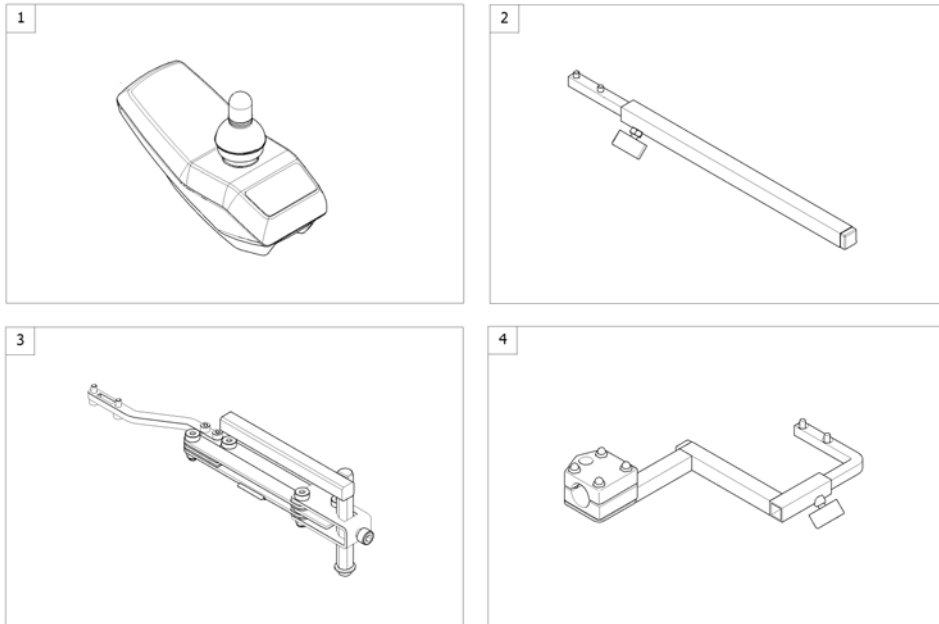

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24SD <i>[1601/1602]</i>		0
2	-	Remote Bracket Standard <i>[1601+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1601+1707/1708]</i>		1
4	-	Remote Bracket Attendant <i>[1602+1207/1208]</i>		1

Remote REM 24SD


142802 8/0

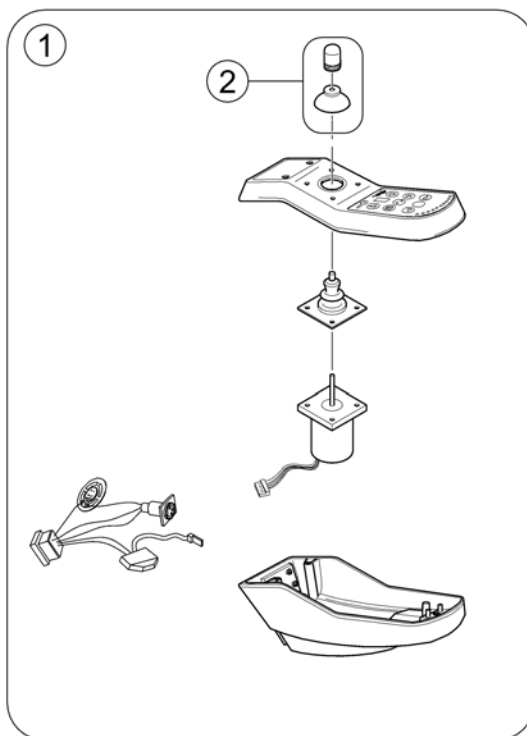

Pos.	Part number	Description	Valid from → until	Qty
1	1436541	Remote REM24SD		1
2	SPZ183229	Remote knob cpl.		1

Remote Bracket Standard


1559851.iso

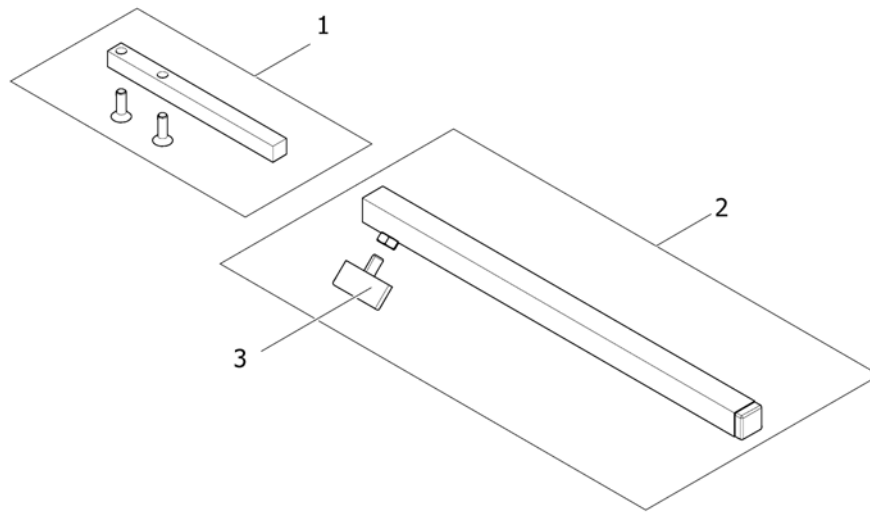

Pos.	Part number	Description	Valid from → until	Qty
1	SP1436542	Support, Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1559852.iso

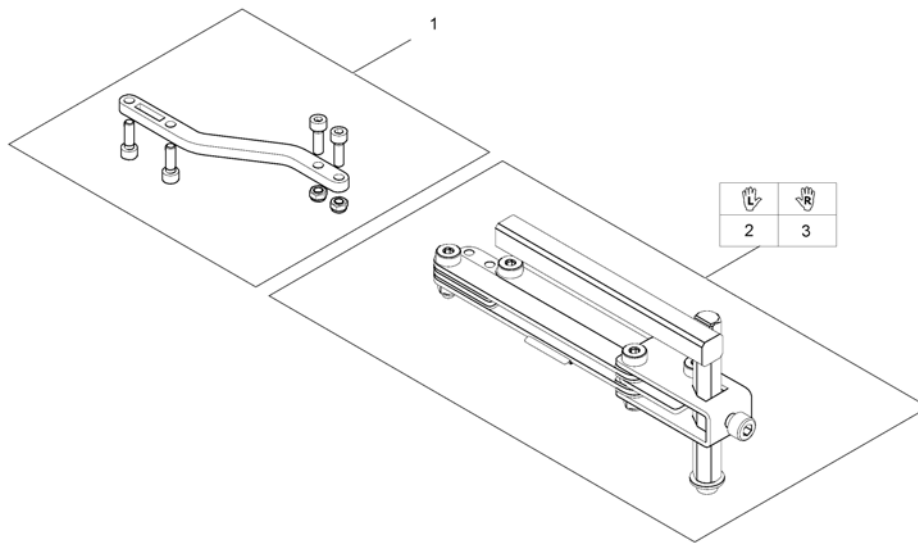

Pos.	Part number	Description	Valid from → until	Qty
1	SP1498222	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Attendant


1560837.00

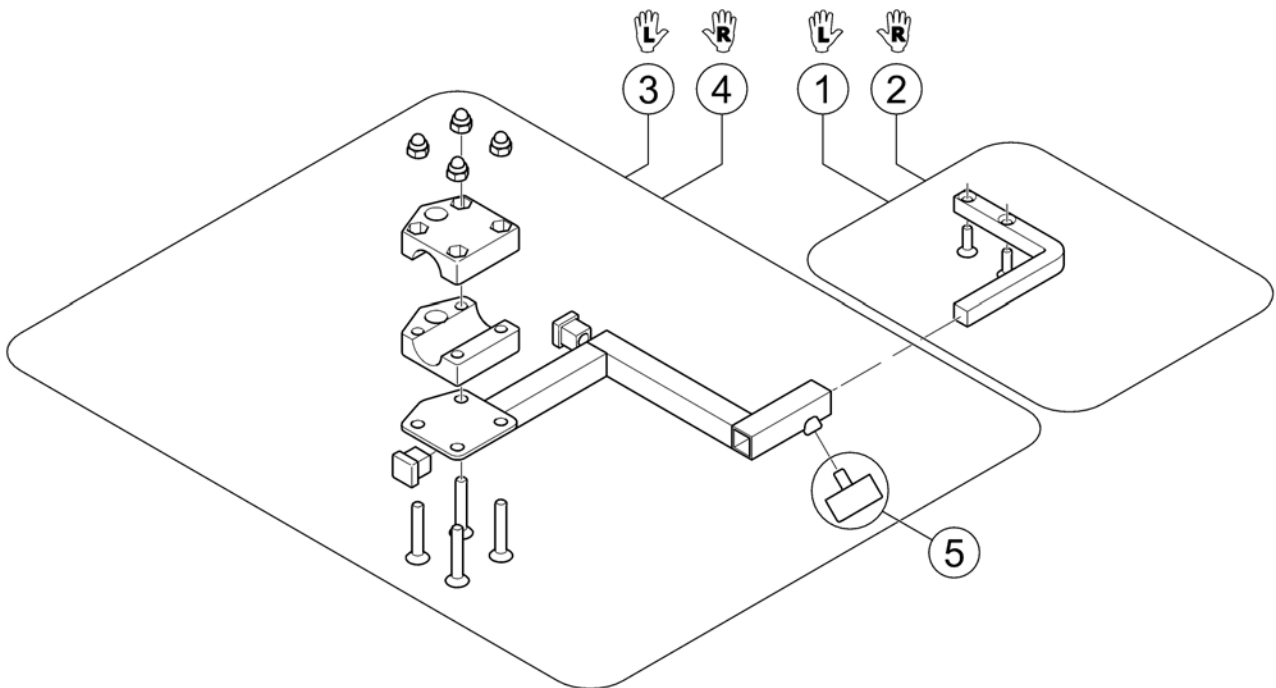

Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1

REM A & B with brackets


1562687.iss

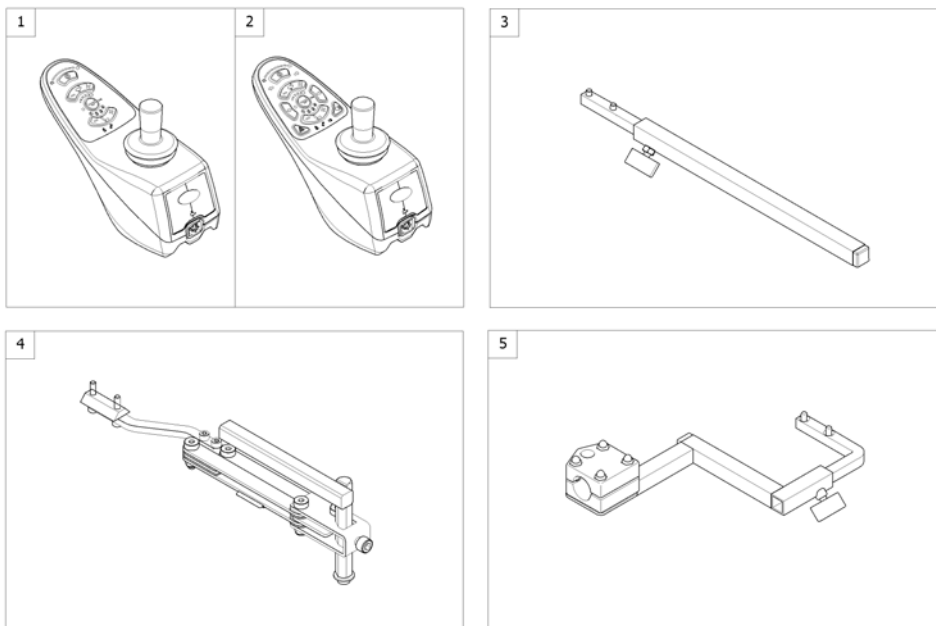

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523440	Remote "REM A" [1638/1643]		1
2	SP1523445	Remote "REM B" [1639/1644]		1
3	-	Remote Bracket Standard [1638/1639+1207/1208]		1
4	-	Remote Bracket Swing away & height adjust. [1638/1639+1707/1708]		1
5	-	Remote Bracket Attendant [1643/1644+1207/1208]		1

Remote Bracket Swing away & height adjust.


1559857.iso

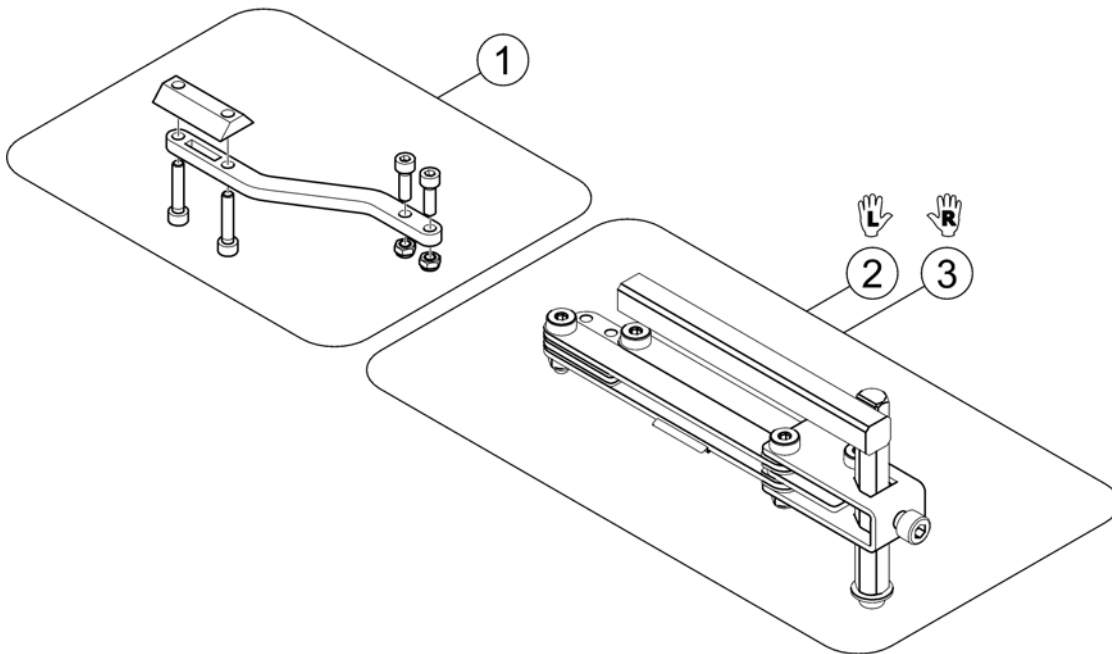

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536180	Support for Remote Bracket, height adjustable & swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

REM 550 with brackets



1562658.iss

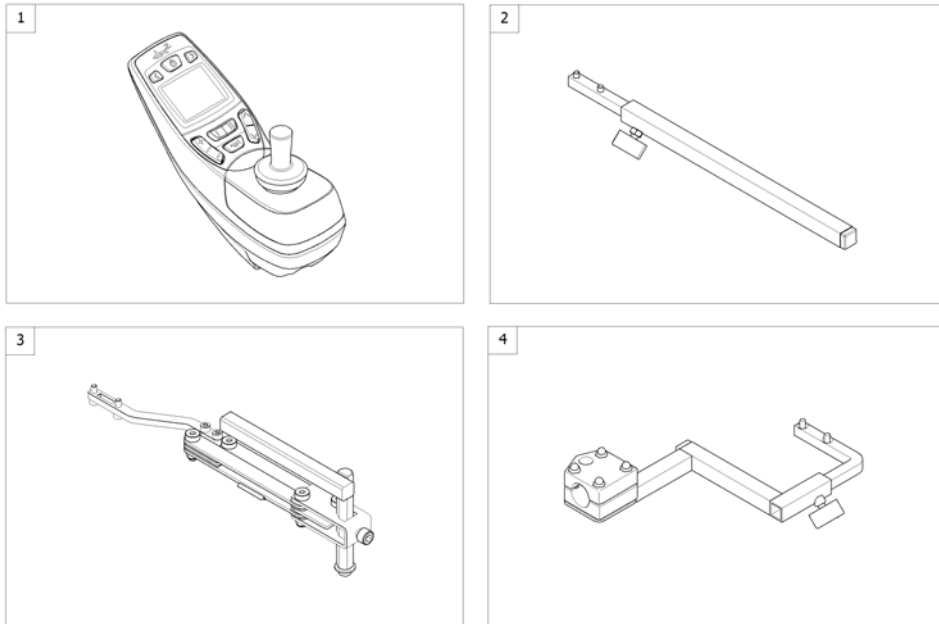

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523444	Remote REM550 <i>[1617/1618]</i>		1
2	-	Remote Bracket Standard <i>[1617+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1617+1207/1208]</i>		1
4	-	Remote Bracket Attendant <i>[1618+1207/1208]</i>		1

REM 34 with brackets


1560846 iso

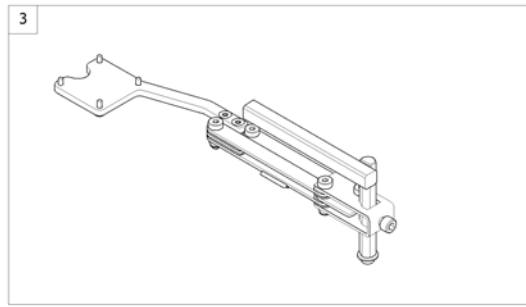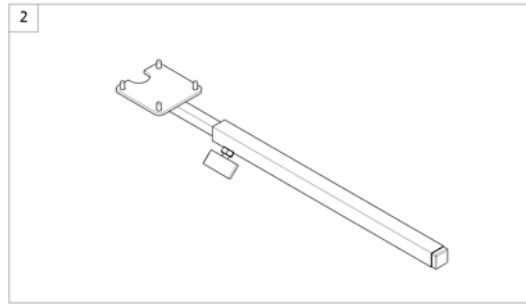

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440677	Remote REM 34 [1611]		1
2	-	Remote Bracket Standard [1611+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1611+1707/1708]		1

Remote Bracket Standard


1560844.iso

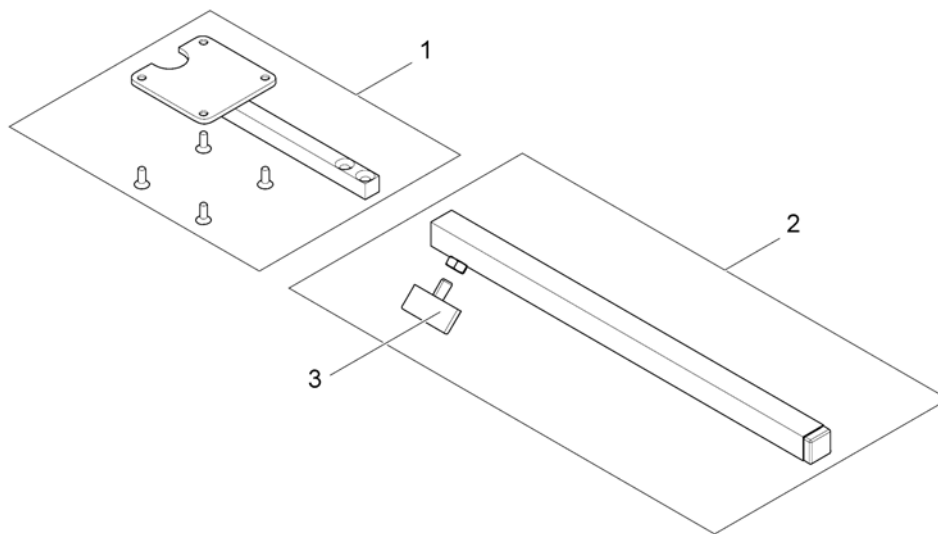

Pos.	Part number	Description	Valid from → until	Qty
1	SP1434921	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1560845.iso

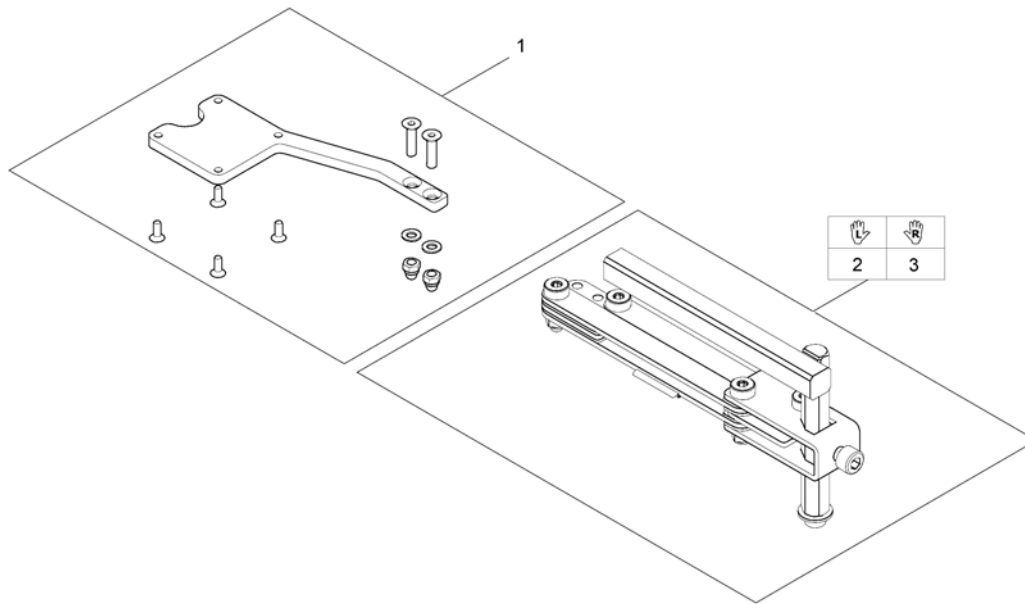

Pos.	Part number	Description	Valid from → until	Qty
1	SP1527125	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

G90A with brackets


1562659.iso

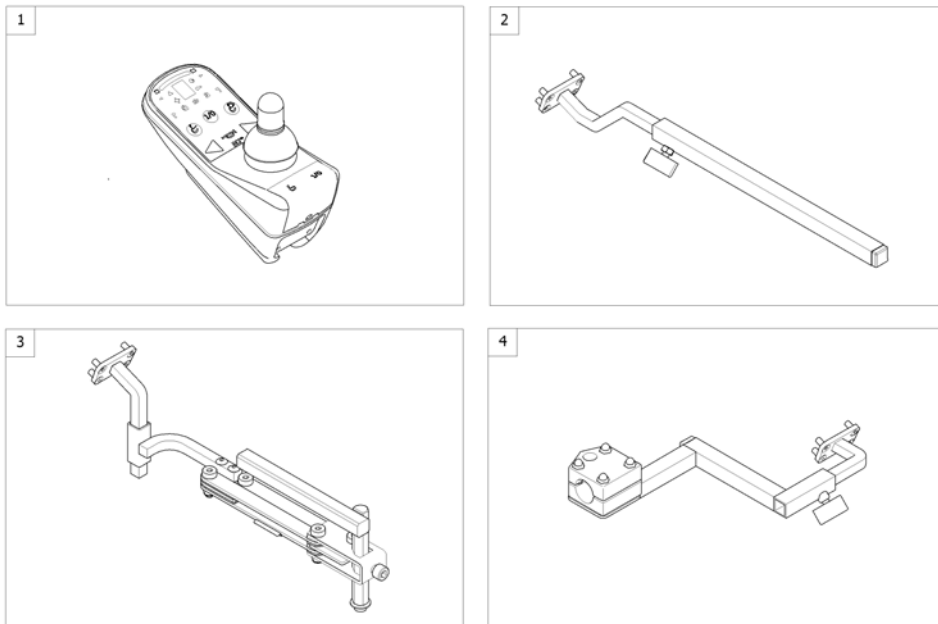

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504720	Remote G90A [1614/1616]		1
2	-	Remote Bracket Standard [1614+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1614+1707/1708]		1
4	-	Remote Bracket Attendant [1616+1207/1208]		1

Remote Bracket Standard


1562939.iso

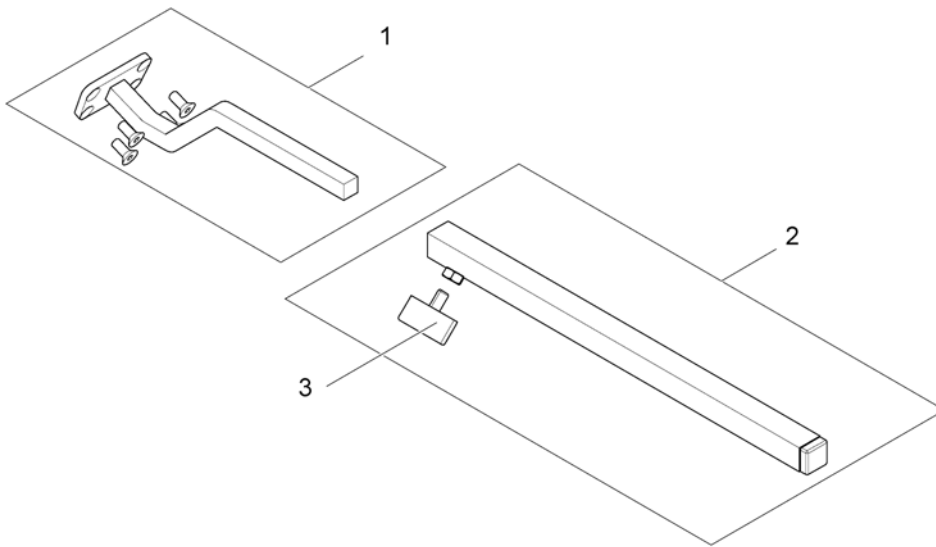

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504822	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

Remote Bracket Swing away & height adjust.


1560310.iso

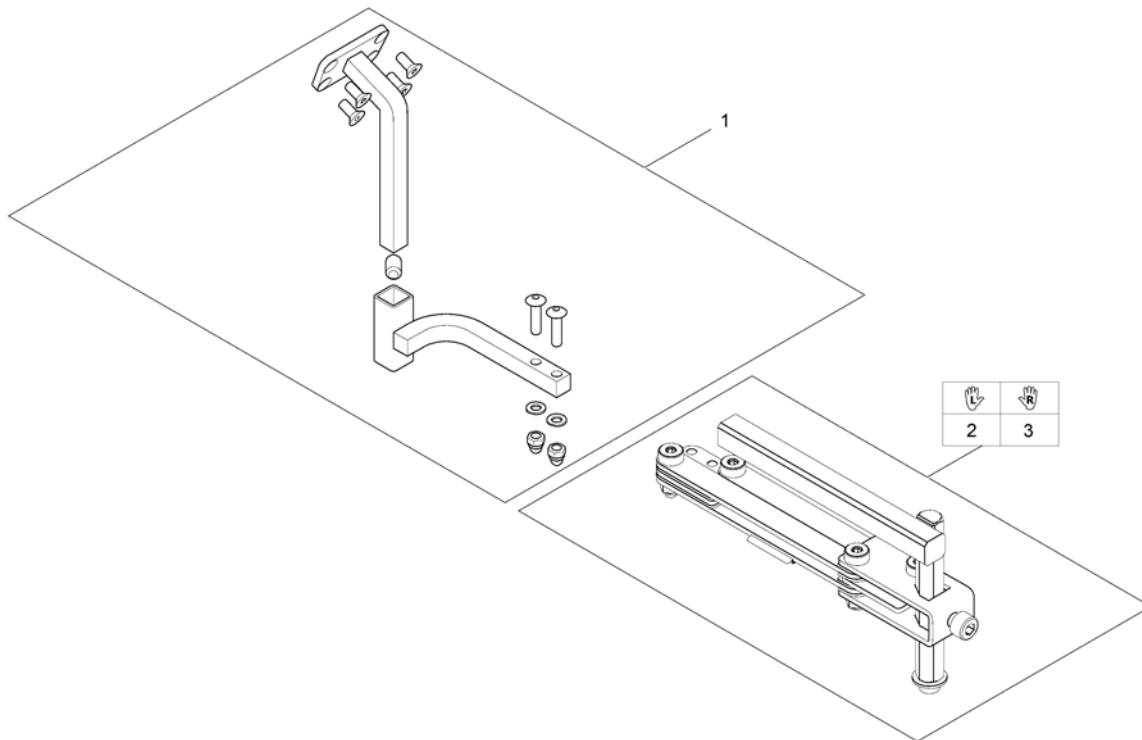

Pos.	Part number	Description	Valid from → until	Qty
1	SP1517352	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

Remote Bracket Attendant


1560853.iso

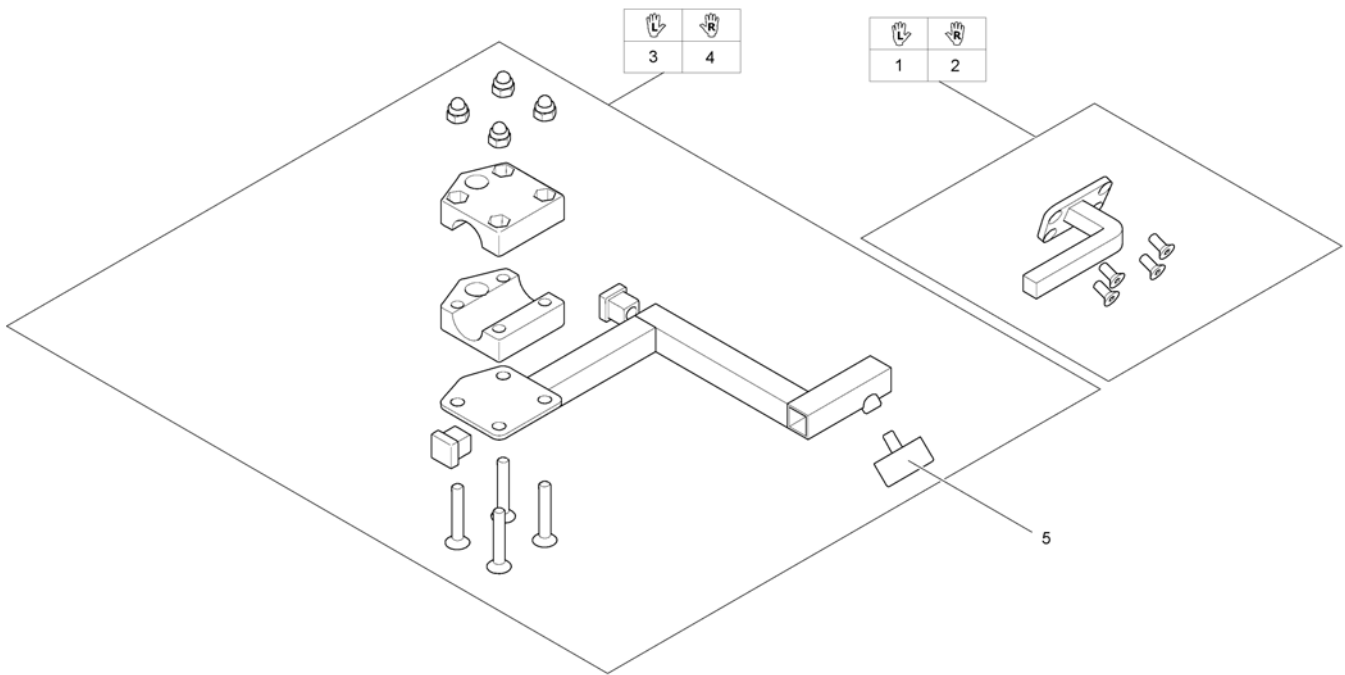

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1

G91S with brackets


1560850 iso

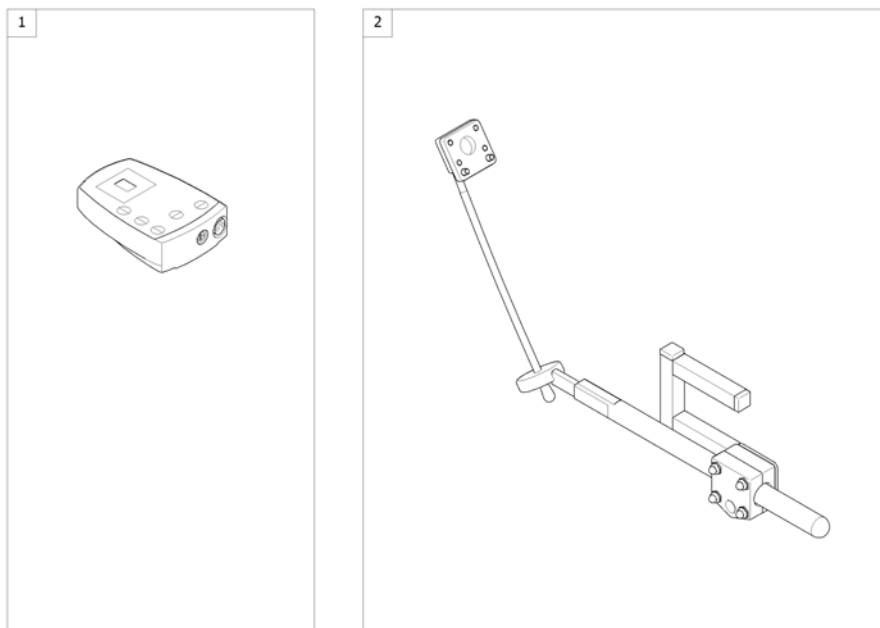

Pos.	Part number	Description	Valid from → until	Qty
1	SP1505187	Remote Control G91S <i>[1615]</i>		1
2	-	Remote Bracket Standard <i>[1615+1207/1208]</i>		1

Remote Bracket Standard


1560654.iso

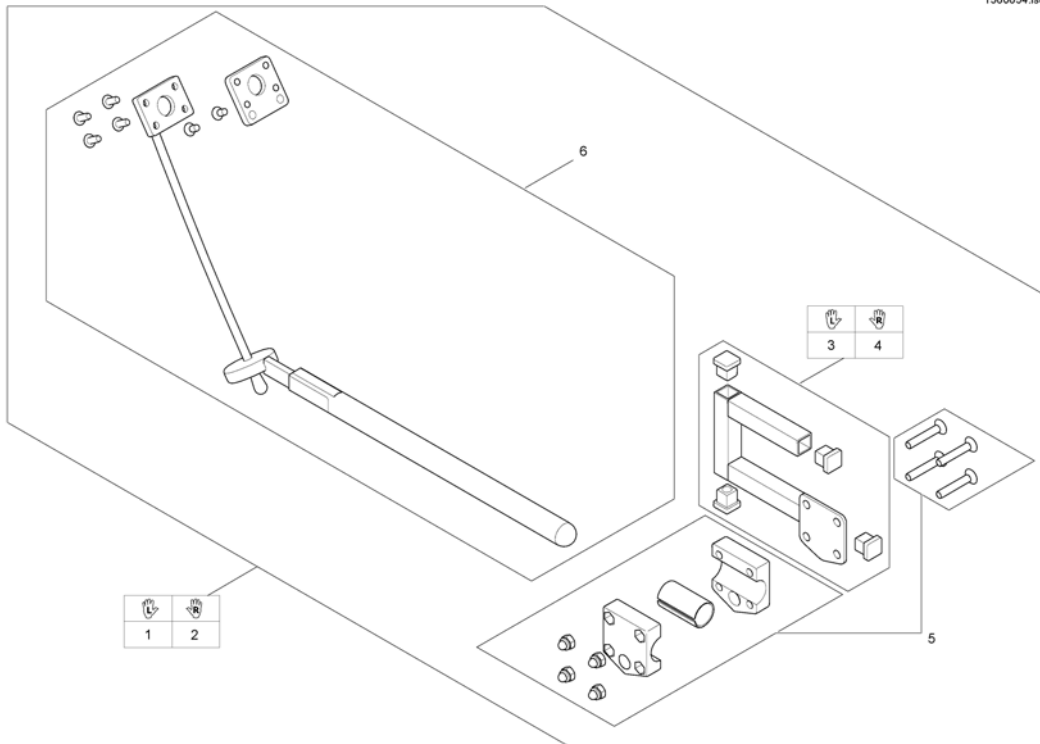

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509718	Remote Bracket G91/G91S LH, cpl		1
2	SP1509690	Remote Bracket G91/G91S RH, cpl.		1
3	SP1509716	Adapter G91 LH		1
4	SP1509711	Adapter G91 RH		1
5	SP1527503	Gripper clamp cpl.		1
6	SP1527504	Swan neck remote bracket with adapter plate		1

Centre control REM 41

F383057.dwg

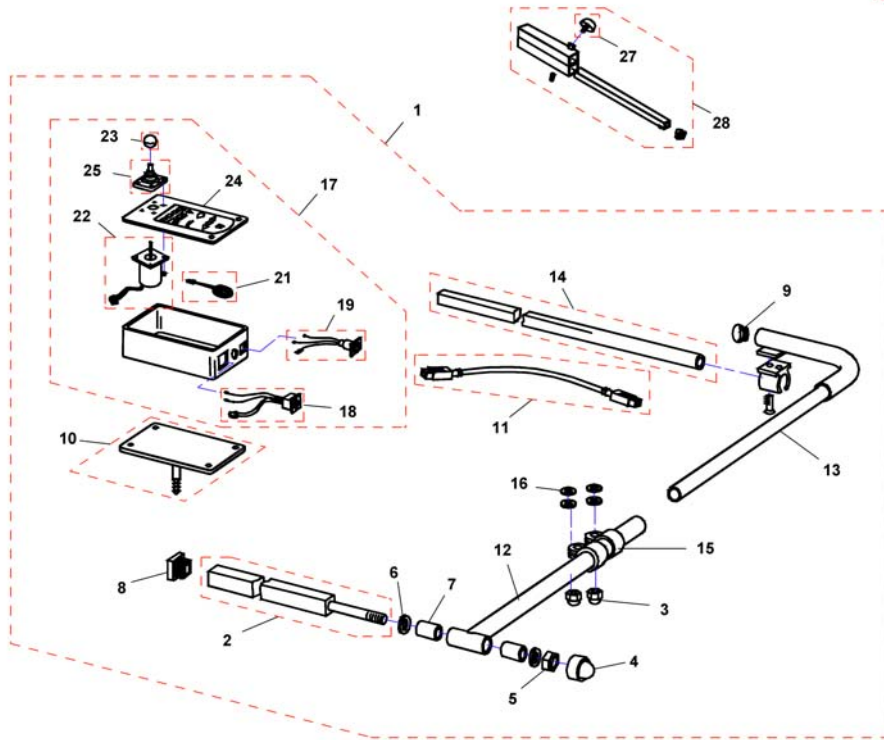


Pos.	Part number	Description	Valid from → until	Qty
-	Q99081	User Manual Table- & Middle Control REM41-S German		1
1	SP1530588	CENTRAL STEERING CPL.; SWING-AWAY RH, REM41-D		1
1	SP1530589	CENTRAL STEERING CPL.; SWING-AWAY LH, REM41-D		1
2	Z177024.00	SUPPORT TUBE; CHROME PLATED		1
3	F60208	Cap nut DIN1587-M08 zc		1
4	L01037	Sks Nut Cap M12		1
5	F20106	Hexagon nut DIN985-M12 zc		1
6	G00104	Washer DIN125-Ø13 zc		1
7	Z17032A	Bush Rd. 19 X 3.5 X 25		1
8	L01055	Plug 15x15mm		1
9	L01008	Tube End Plug SI 22		1
10	Z183216.P40	SUPPORT BRACKET; RAL 9005 BLACK		1
11	1559862	Bus cable ACS, 1700mm		1
12	Z183089.00	SWING ARM; CHROME PLATED		1
13	Z183086.00	SUPPORT ARM RH; CHROME PLATED		1
13	Z183092.00	SUPPORT ARM LH; CHROME PLATED		1
14	H281005A.00	MIRROR SUPPORT RAIL; CHROME PLATED		1
15	5006720	Klemmschelle F. Dm 22 mm		1
16	G00106	Washer DIN125-A-Ø8,4 zc		1
17	SP1527930	DX ACS Tray Remote DX-REM41		1
18	P41108	ACS-BUS CABLE SOCKET		1
19	P41106	CHARGER SOCKET		1
21	P30493	HORN		1
22	GSM55120	ACS-remote WITH 180MM CABLE		1
23	L36999	KNOB FOR remote 32 MM, HOLE 5.8 MM		1
24	-	No longer available.		0
25	Z183001A	Rubber gaiter for ACS remote		1
26	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1
27	L31001	Wing ScREW M6X10		1
28	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1

Small Chin Control with brackets


Pos.	Part number	Description	Valid from → until	Qty
-		Small Chincontrol		1
-		Framework for chincontrol		1

Small Chincontrol

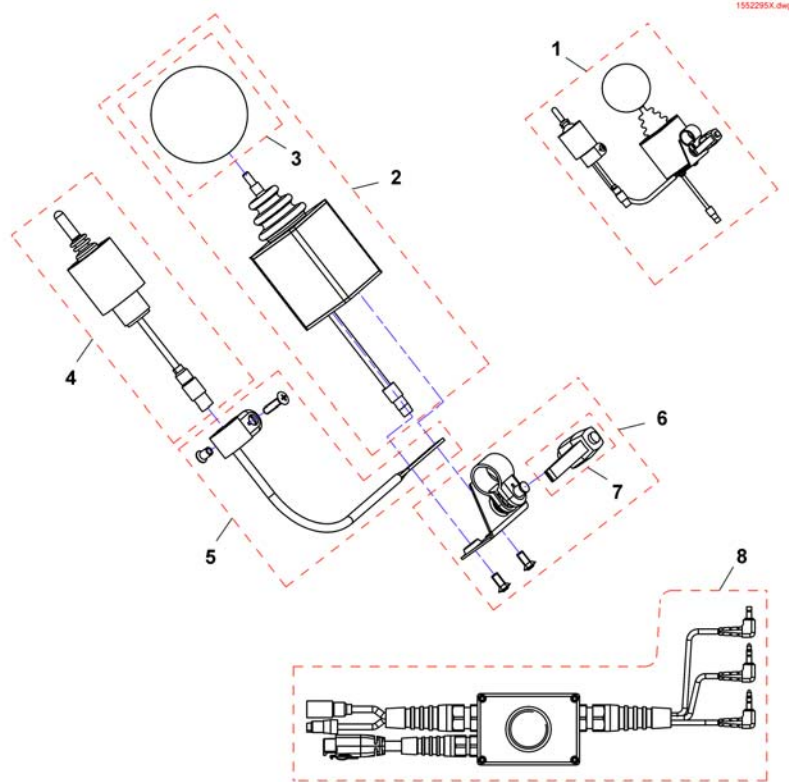

Pos.	Part number	Description	Valid from → until	Qty
1	1448705	Chin control REM24, cpl.		1
1	1421770	Chin control G80/G90, cpl.		1
1	SP1544255	Chin control G91, cpl.		1
2	Z183282	Joystick, kmpl.		1
3	Z183265	Foam control knob cpl.		1
4	1442943	Switchjoystick REM24, kmpl.		1
4	Z183283	Switchjoystick G80/G90, cpl.		1
4	SP1544250	Switchjoystick G91, kmpl.		1
5	SP1552294	Switchjoystick bracket		1
6	SP1552295	Joystick clamp		1
7	L32901	Clampinglever		1
8	SP1442944	Harness REM24		1
8	Z183275	Harness G80/G90		1
8	SP1541715	Harness G91		1




Framework for chincontrol

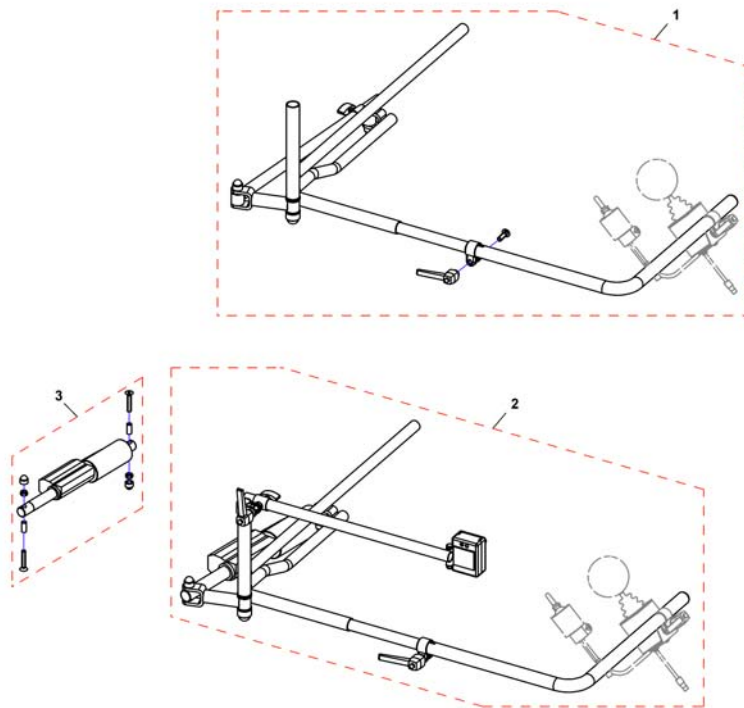

Pos.	Part number	Description	Valid from → until	Qty
-	Z167994	CHIN CONTROL SUPPORT . CPL. <i>Not shown / Nicht abgebildet / Non illustré</i>		1
1	SP1422943	Framework for chin control, manual, horizontal swing away, LH, cpl.		1
1	SP1422942	Framework for chin control, manual, horizontal swing away, RH, cpl.		1
2	SP1422945	Framework for chincontrol, powered, horizontal swing away, LH, cpl. <i>incl. Actuator</i>		1
2	SP1422944	Framework for chin control, powered, horizontal swing away, RH, cpl. <i>incl. Actuator</i>		1
-	SP1552335	Hardware, chin control, powered, horizontal swing away, LH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1552336	Hardware, chin control, powered, horizontal swing away, RH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1422946	Framework for chincontrol, powered, vertikal swing away <i>Not shown / Nicht abgebildet / Non illustré</i>		1
3	SP1552337	Actuator for horizontal swing away chin control, cpl.		1
3	SP1552418	Gestell für Kinnbedienung, elektrisch, vertikal abschwenkbar <i>Not shown / Nicht abgebildet / Non illustré</i>		1

Remote Dual Control with Brackets


1574292 iso

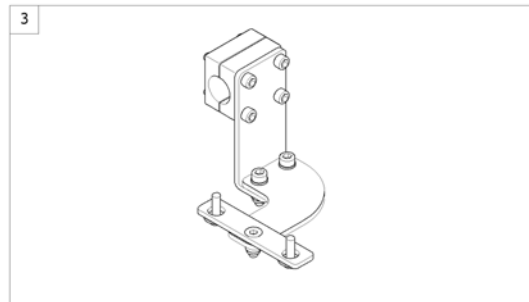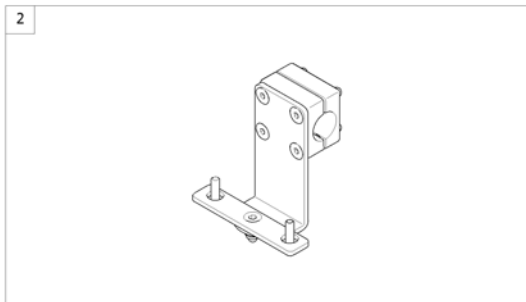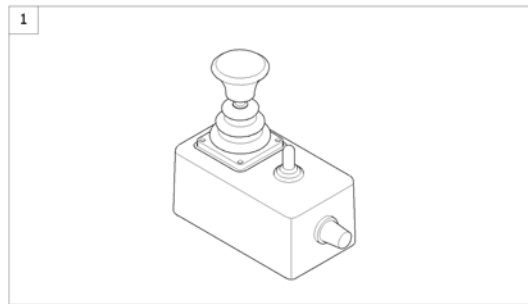

Pos.	Part number	Description	Valid from → until	Qty
1	6001425	ACS dual control <i>[1291/1292]</i>		1
2	-	Remote Bracket Dual Control for Standard-/Contour-/Flex-/Junior-Seat		1
3	-	Remote Bracket Dual Control for Modulite Seat		1

Remote Bracket Dual Control for Standard-/Contour-/Flex-/Junior-Seat


1560885.iso

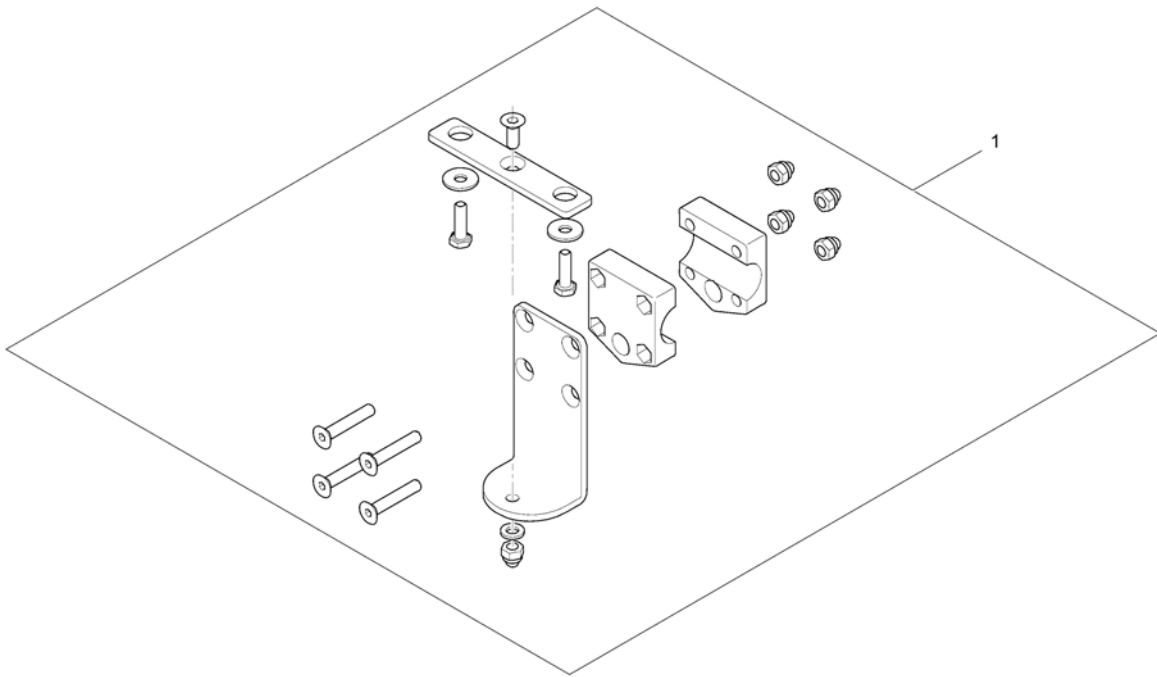


Pos.	Part number	Description	Valid from → until	Qty
1	SP1422818	Remote Bracket Dual Control		1

Remote Bracket Dual Control for Modulite Seat

1574291 iso

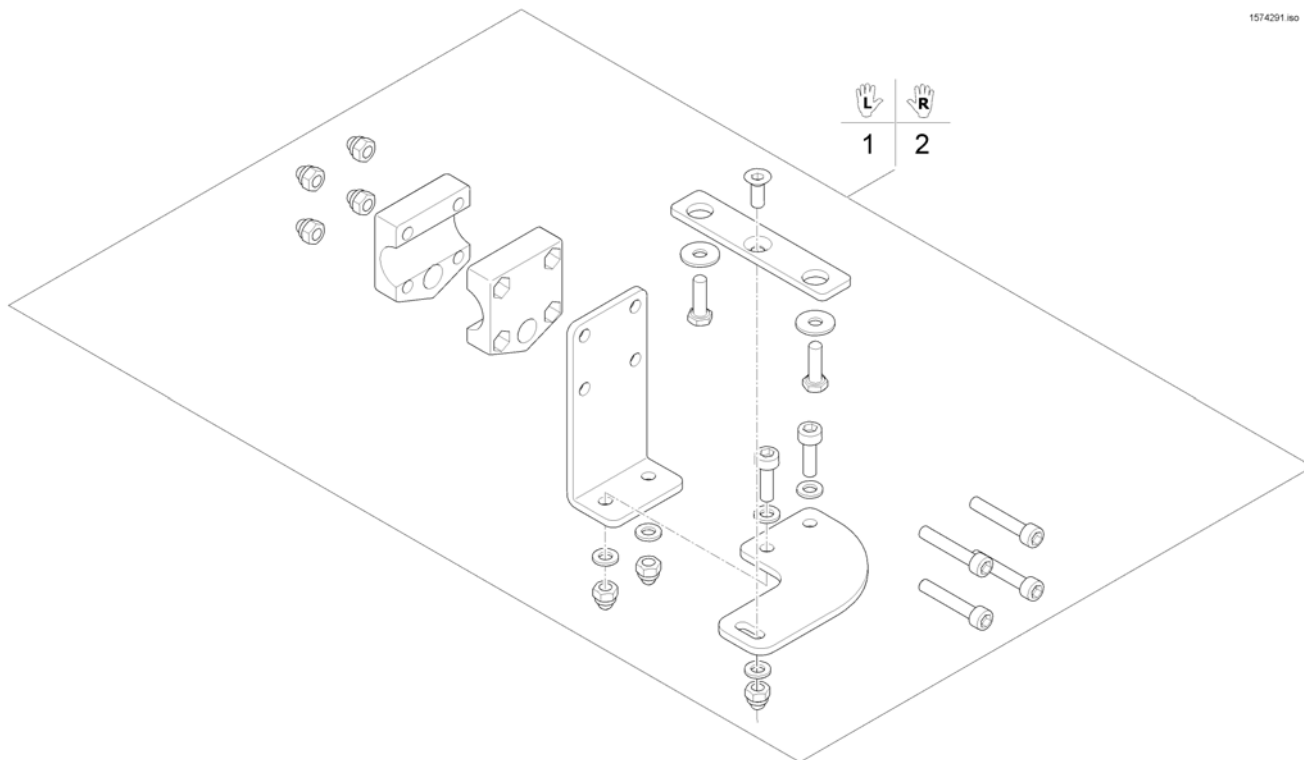

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571799	Remote Bracket Dual Control LH		1
2	SP1571800	Remote Bracket Dual Control RH		1

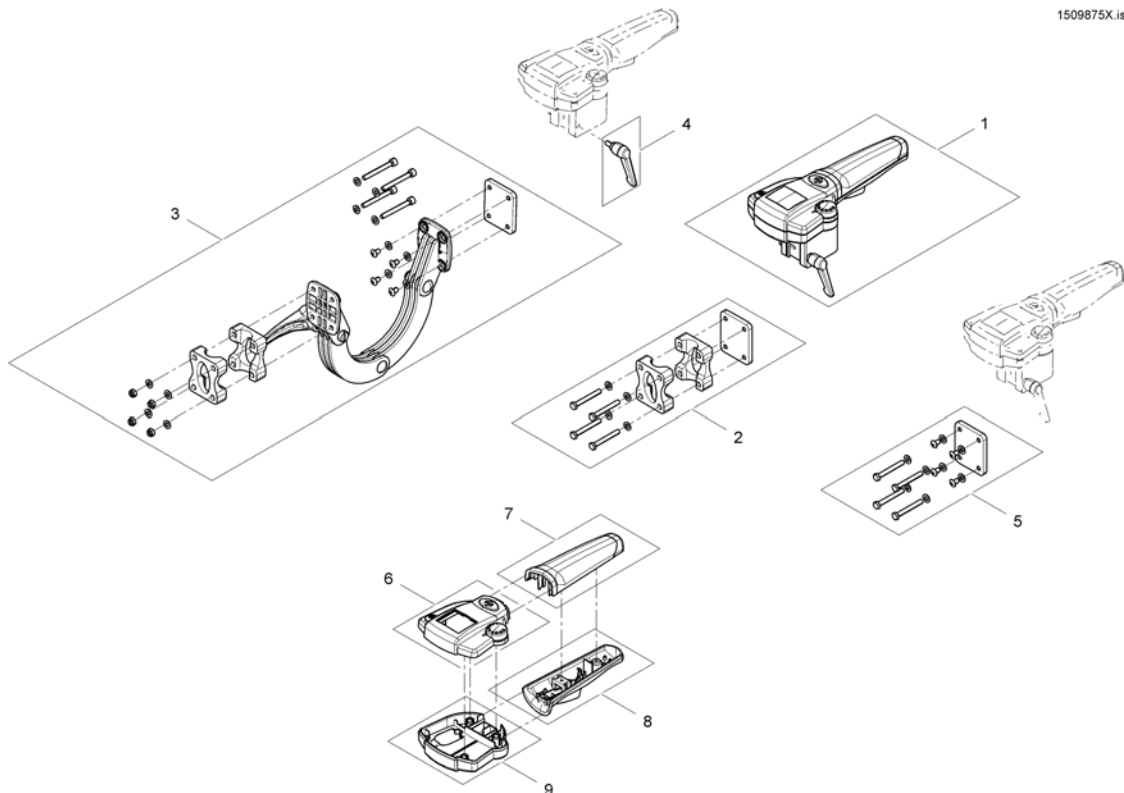


Pos.	Part number	Description	Valid from → until	Qty
1	SP1509875	Intuitiv Dual Control (IDC) <i>For chairs build before 01/02/10 please also order SP1537148</i>		1
2	SP1522310	Fixed bracket		1
3	SP1521488	Adjustable bracket		1
4	SP1510135	Clamp lever M5x10		1
5	SP1537148	Supportplate IDC for chairs before 01.02.10		1
6	SP1579573	Upper Housing		1
7	SP1511060	Handhold upper part		1
8	SP1511099	Handhold lower part		1
9	SP1510470	Housing lower part		1

ACS Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables ACS		1
-		T-Adapter		0
-		T-Adapter - Modulite Seat		1
-		Programming Kits		1
-		Magnetkey for Remotes		1

Bus-Cables ACS

ACSBUSKABEL.iso

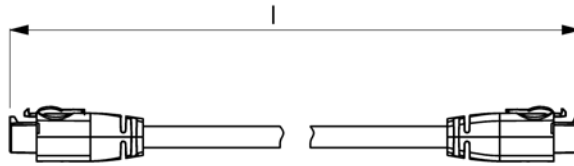


Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060

Pos.	Part number	Description	Valid from → until	Qty
1	GSM63003	Bus cable ACS, 300mm		1
2	SPGSM63005	Bus cable ACS, 500mm <i>Replace GSM63004</i>		1
3	GSM63007	Bus cable ACS 700mm		1
4	GSM63011	Bus cable ACS, 1100mm		1
5	1559862	Bus cable ACS, 1700mm <i>Replace GSM63016</i>		1
6	GSM63020	Bus cable ACS, 2000mm <i>Replace 1419386</i>		1
7	1440424	Bus cable ACS, 2500mm		1
8	SP1559873	Bus cable ACS, 2700mm <i>Replace GSM63026</i>		1
9	SP1550402	Extension cable ACS, 1060mm		1

T-Adapter


1583783 iso

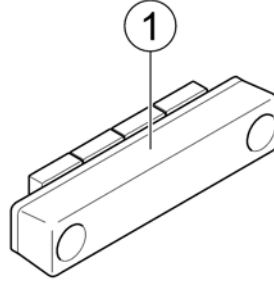

Pos.	Part number	Description	Valid from → until	Qty
1	P10072	ACS-Bus T-Adapter (Splitter)		1

T-Adapter - Modulite Seat


1583360 iso

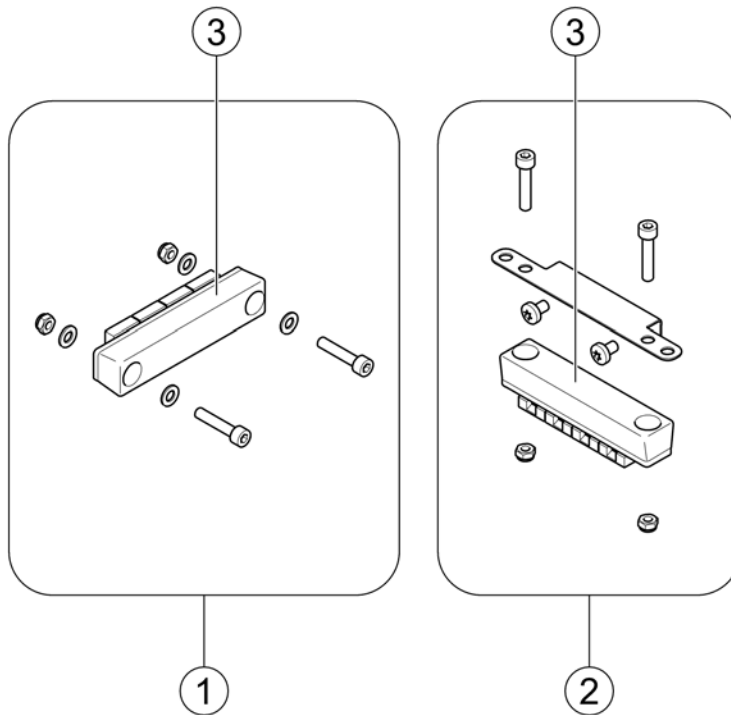

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571826	T-Adapter - Modulite Seat		1
2	SP1569958	T-Adapter - Modulite Seat with MBL-Modul		1
3	P10072	ACS-Bus T-Adapter (Splitter)		1

Programming Kits


1584805 iso

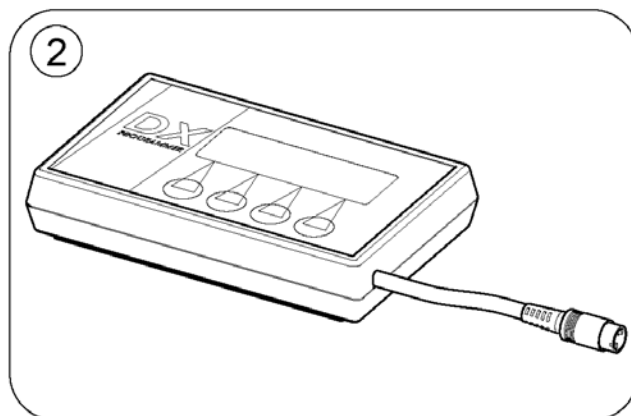

Pos.	Part number	Description	Valid from → until	Qty
1	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	6001450	Hand Programmer, Instrucions In French, German, Spanish		1
	SP1441551	Programming Adapter Shark DK-ADAPT (necessary to program Shark)		1

Magnetkey for Remotes


1564186.iso

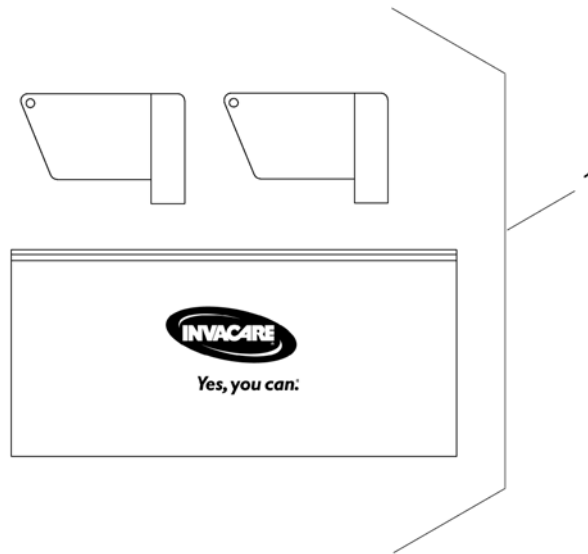

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419695	Magnet key with bag		1

Shark Electronic

1500784.sp

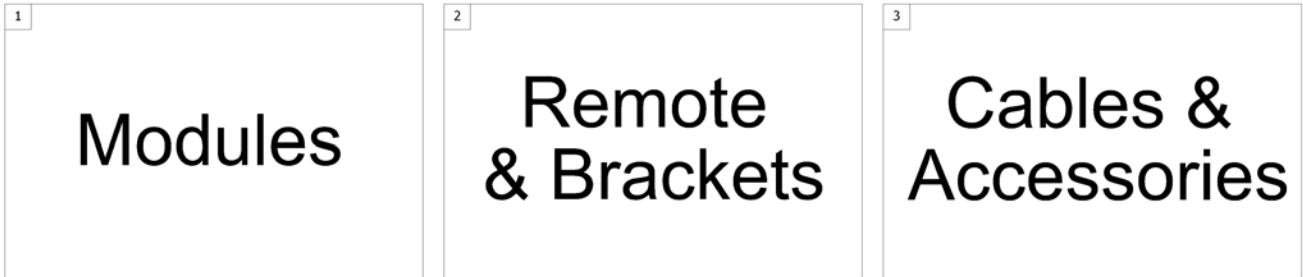


Pos.	Part number	Description	Valid from → until	Qty
1	-	Shark Electronic Modules		1
2	-	Shark Remote & Brackets		1
3	-	Shark Cables & Accessories		1

Shark Electronic Modules

Pos.	Part number	Description	Valid from → until	Qty
-		Shark Powermodule		1
-		Lightboard ACS2		1


Shark Powermodule

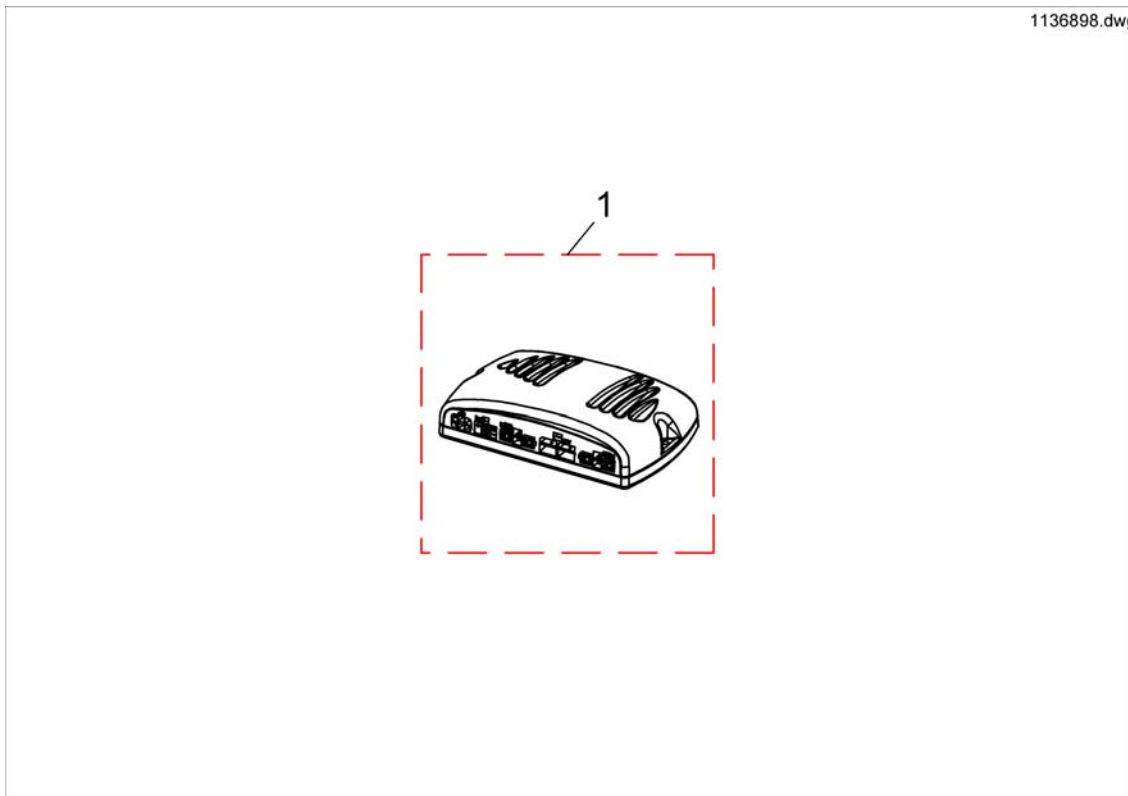

Pos.	Part number	Description	Valid from → until	Qty
-	1450313	Buscable Shark 1500mm		1
1	1136898	Powermodule, Shark, 90A, 2 actuators, DK-PMC28		1

Shark Remote & Brackets


1560925 iso

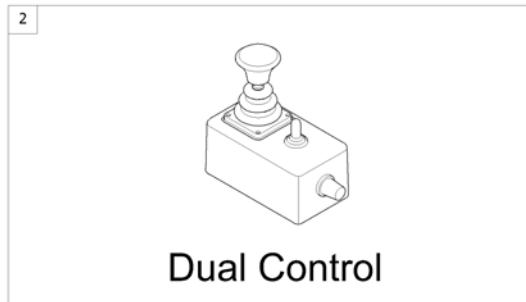



Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote Shark with Brackets <i>[1801]</i>		1
2	-	Remote Dual Control with Brackets <i>[1891/1892]</i>		1

Remote Shark with Brackets

1563326.iss

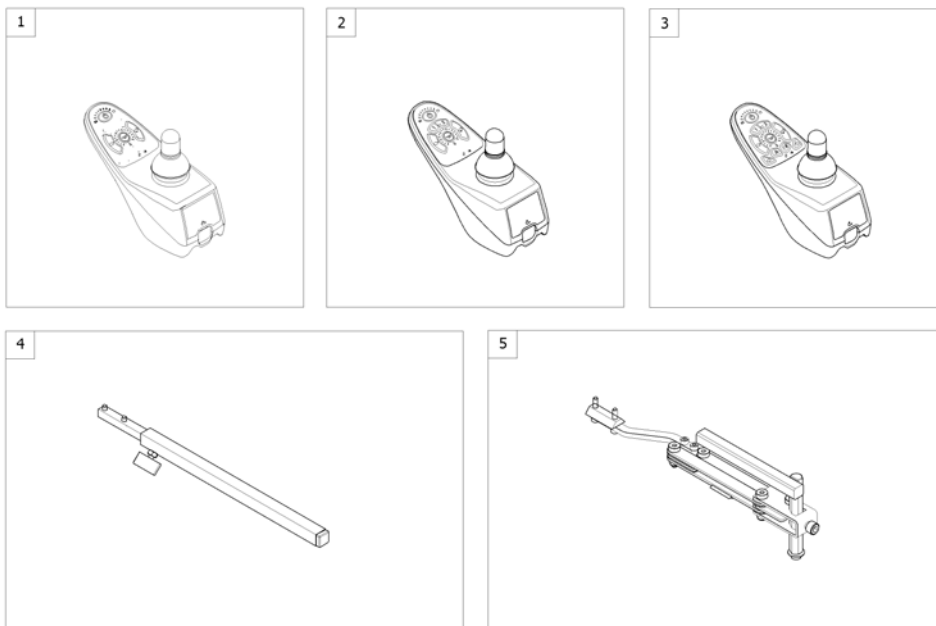


Pos.	Part number	Description	Valid from → until	Qty
1	1492567	Remote DK-REM D01, Shark2 (Drive only) [1801]		1
2	1449177	Remote DK-REM D11, Shark2 (Drive & Adjustment) [1801]		1
3	1484904	Remote DK-REM D21, Shark2 (Drive, Adjustment, Light) [1801]		1
4	-	Remote Bracket Standard [1801+1207/1208]		1
5	-	Remote Bracket Swing away & height adjust. [1801+1707/1708]		1

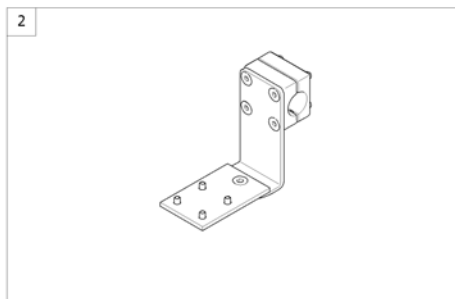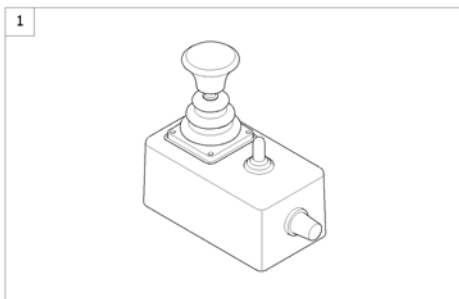

Pos.	Part number	Description	Valid from → until	Qty
1	1444717	Shark Dual Control, switchable <i>[1891/1892]</i>		1
2	-	Remote Bracket Dual Control <i>[1891/1892]</i>		1

Remote Bracket Dual Control

1560883.iso

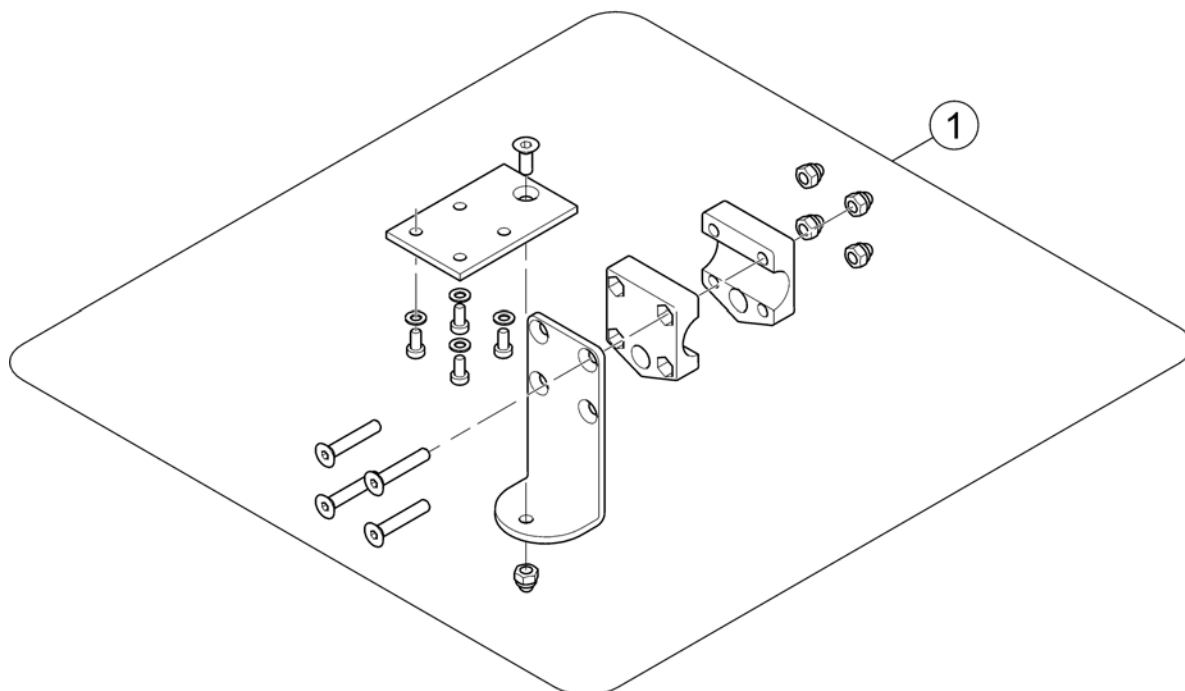


Pos.	Part number	Description	Valid from → until	Qty
1	1450307	Remote Bracket Attendent Control Shark		1

Shark Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables Shark		0
-		Cables Shark		0
-		Programming Kits		0

Bus-Cables Shark


SHARKBUSKABEL.00

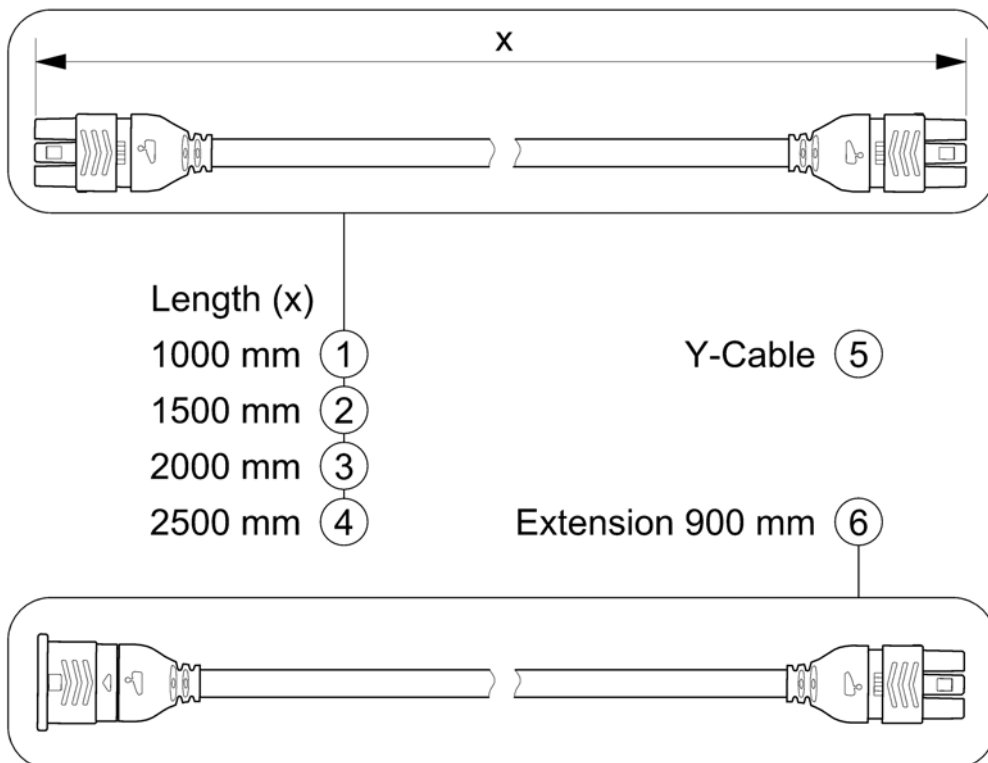

Pos.	Part number	Description	Valid from → until	Qty
1	1448922	Bus Cable Shark 1000mm		1
2	1450313	Buscable Shark 1500mm		1
3	1484911	Buscable Shark 2000mm		1
4	SP1510073	Buscable Shark 2500mm		1
5	1450314	Y-Cable		1
6	1116404	Extension Cable Shark 900mm		1

Cables Shark


1583705.000

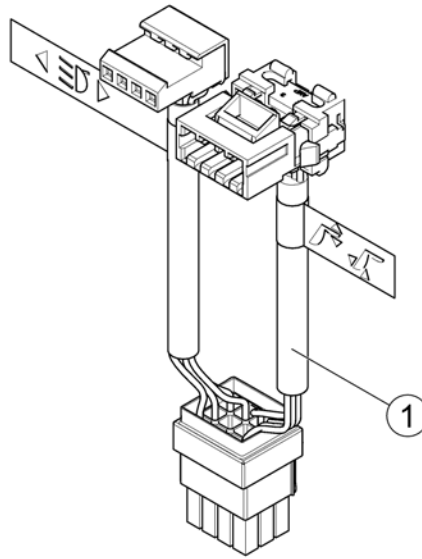

Pos.	Part number	Description	Valid from → until	Qty
1	1488951	Wire Harness Shark - electrical adjustments & light		1

Chargers

1589799.iso

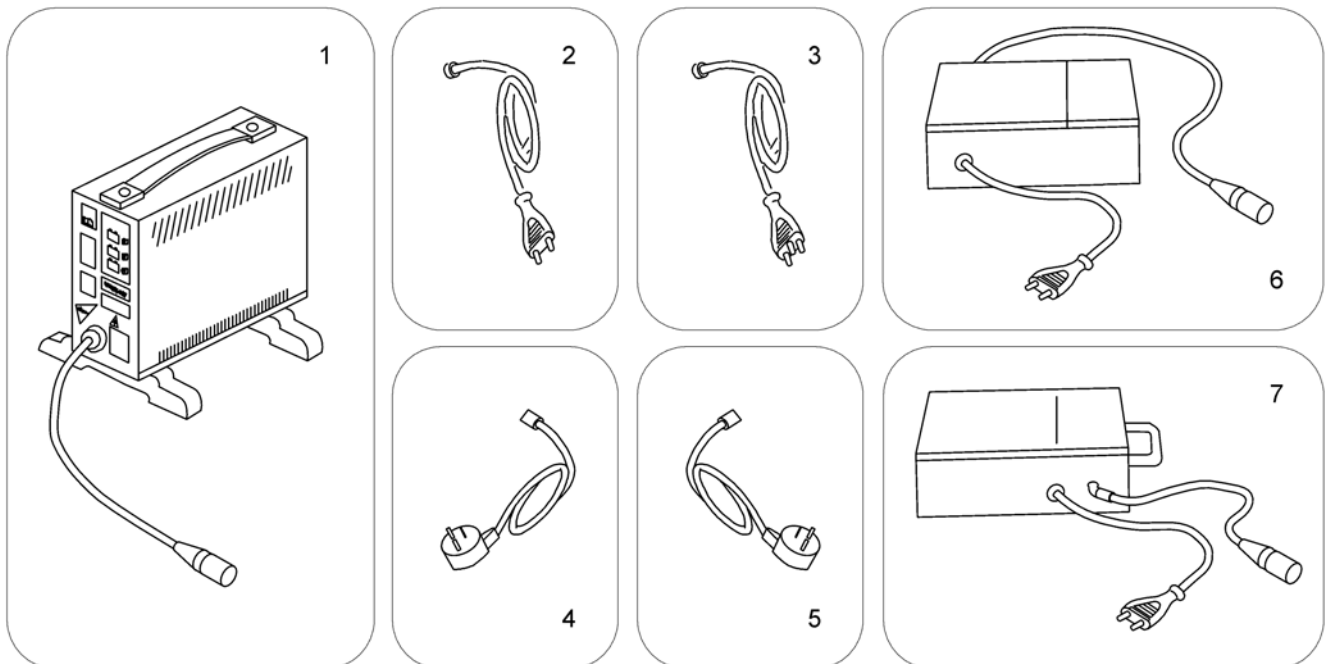

Pos.	Part number	Description	Valid from → until	Qty
1	SPP51981	Charger 8A without Power Cord <i>[1343]</i>		1
2	P21079	Power Cord Continental (Schuko GCB-0401)		1
3	P21080	Power Cord UK (GCB-0403)		1
4	1443841	Power Cord Swiss (black 6A/250V)		1
5	SP1553849	Power Cord Israel (IEC 320 C13 to SI-32)		1
6	SP1545511	Charger 10A incl. Power Cord Continental <i>[1389]</i>		1
7	5813987	Charger 8A (Protectionclass IP54) incl. Power Cord Continental (Sweden only) <i>[1388]</i>		1


Batteries

	Capacity [Ah]															
	1,2	12	30	31	32	36	38	40	40	50	50	50,9	60	73,6	75	97,6
Bora/SpectraXTR								8		10						
Dragon										10			13			
Dragon Vertic													13			
FDX													13			
G50													13	14		16
Kite													13	14		
Mirage				4				8								
Mistral3				4									12			
P9000XDT					5											
Pronto M41					5											
Pronto M61					5											
Spectra Blitz								8					12			
Spectra Comfort								7								
Spectra Plus								7								
Spectra Seat Riser								7								
Storm3										10			13	14		
Storm3 True Track	1													14		
Storm3 Vertic														14		
Storm4														14		
Storm4 Max														14		
Stream										10			13			
TDX-SP														14		
TDX-SP Narrow													12			
Tiger										10			13			
Topan													13			
Tornado								8								
Typhoon													13			
Typhoon GB	1												13			
Typhoon GB	1												13			
Zephyr	1												13			
Auriga			3					8		10						
Comet														14	15	
Leo						6			9							
Lynx		2														
Meteor														14		
Orion												11				
length [mm]	97	152	197	196	211	196	210	197	198	198	198	238	259	260	260	329
width [mm]	49	99	132	132	132	130	175	168	165	166	165	140	169	171	168	171
height [mm]	51	102	180	183	183	179	175	175	171	176	170	235	178	210	223	238
weight [kg]	0,6	3,9	11,1	10,5	11	12	14,6	14,4	14,2	14	16	17,1	19,3	24,3	26	32,5





Pos.	Part number	Description	Valid from → until	Qty
i	-	Change batteries only pairwise and from the same type!		1
1	1440386	Battery, 12V/1,2Ah C20, GEL		1
2	SP1506493	Battery 12V/12Ah AGM		1
3	P50021	Battery, 12V/30Ah C20, GEL		1
4	P50029	Battery, 12V/31Ah C20, GEL		1
5	SP1521459	Battery, 12V/34Ah C20, AGM		1
6	1505498	Battery, 12V/36Ah C20, AGM		1
7	P50028	Battery, 12V/38Ah C20, GEL (UK & DK only)		1
8	PG50988	Battery, 12V/40Ah C20, GEL		1
9	SP1550692	Battery, 12V/40Ah C20, AGM		1
10	SP1522107	Battery, 12V/50Ah C20, AGM		1
11	SP1521695	Battery, 12V/50Ah C20 AGM		1
12	PG50989	Battery, 12V/50,9Ah C20, GEL		1
13	1432944	Battery, 12V/60Ah C20, GEL		1
14	P50016	Battery, 12V/73,6 Ah C20, Gel		1
15	SP1522026	Battery, 12V/75Ah C20, AGM		1
16	1478675	Battery, 12V/97,6Ah C20, GEL		1

Battery support

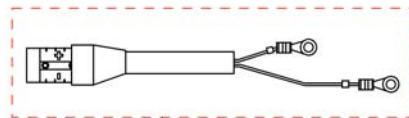


2

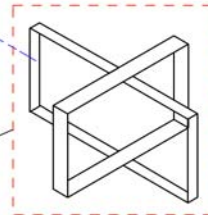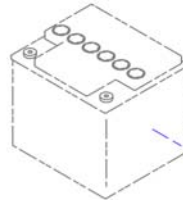

1


Pos.	Part number	Description	Valid from → until	Qty
	1150940	Batteryharness for Tilt with Recline and Lighting		1
	GSM63003	Bus cable ACS, 300mm		1
	1432561	Strip Fuse Holder with 80A Fuse		1
1	F384020	Lashing strap		1
2	SP1527780	Battery Cable ACS2		1

Battery Tray Assembly and Wire Harnesses

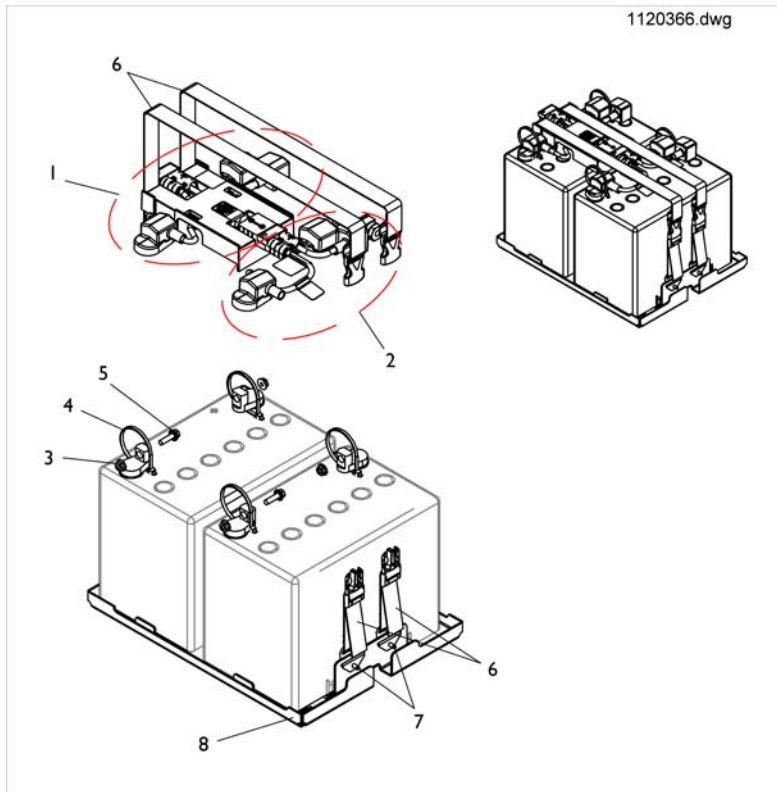

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523007	Battery Wire Harness with Terminal Hardware, GP24 -Rear		1
2	SP1523006	Battery Wire Harness with Terminal Hardware, GP24 -Front		1
4	SP1522970	Package, Cable Ties (11-1/2"Long)		10
5	SP1522972	Screw, Hex Flange Whiz Lock (1/4-20x7/8")		1
6	SP1523191	Battery Strap Assembly, GP24/3-22NF		1
7	SP1523266	Rivet (3/16 x7/16")		1
8	SP1523188	Battery Tray Assembly, TDX SP GP24		1
	SP1523150	Strip, Self Adhesive Wear (2-1/2'Long x1/2"Wide)		1


Light System

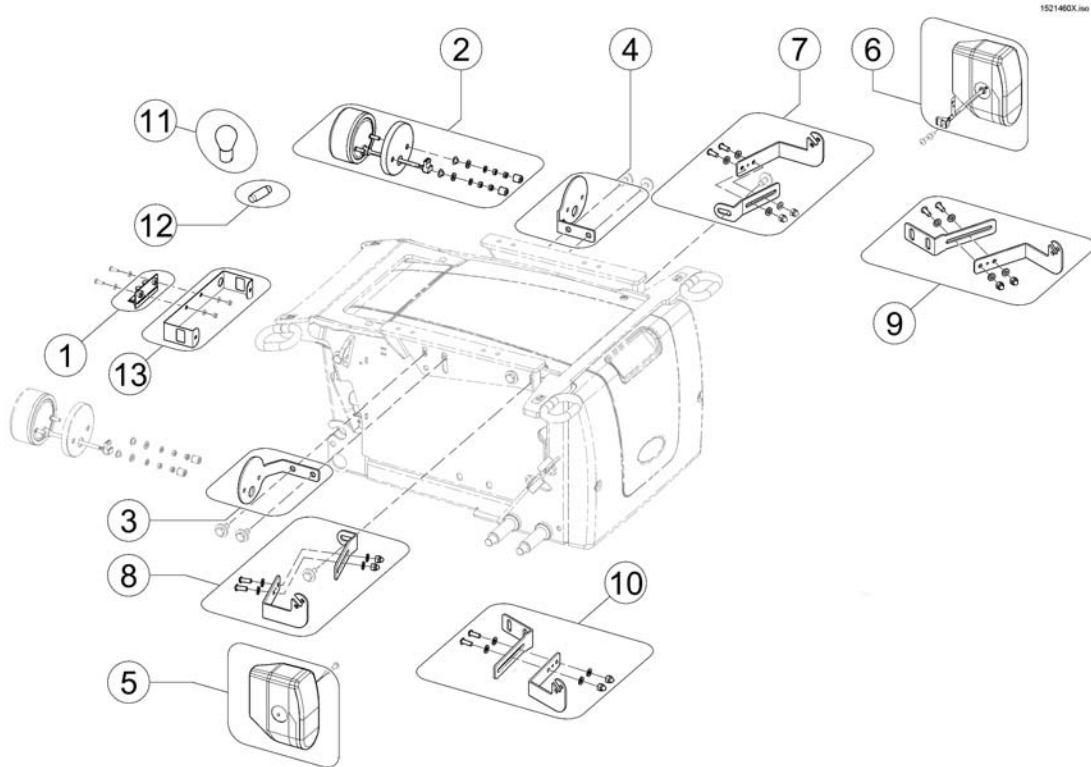

Pos.	Part number	Description	Valid from → until	Qty
-	1488951	Wire Harness Shark - electrical adjustments & light		1
1	1415004	Central Light Distributor Circuit Board ACS/Shark		1
2	1442736	Rear Light 24V, cpl.		1
-	-	Rear Light Protector Kit		0
3	SP1513692	Rearlight bracket RH		1
4	SP1513691	Rearlight bracket LH		1
5	1420412	Headlight 24V, RH, cable 900mm, without bracket		1
6	1420413	Headlight 24V, LH, cable 900mm, without bracket		1
7	SP1513751	Headlight bracket front LH		1
8	SP1513752	Headlight bracket front RH		1
9	SP1551039	Headlight bracket LH - Narrow Base		1
10	SP1551040	Headlight bracket RH - Narrow Base		1
11	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
12	SPP31191	Incandescent Lamp 24V/5W white		1
13	SP1513830	Bracket for light connector		1

Rear Light Protector Kit


1585420 (6)

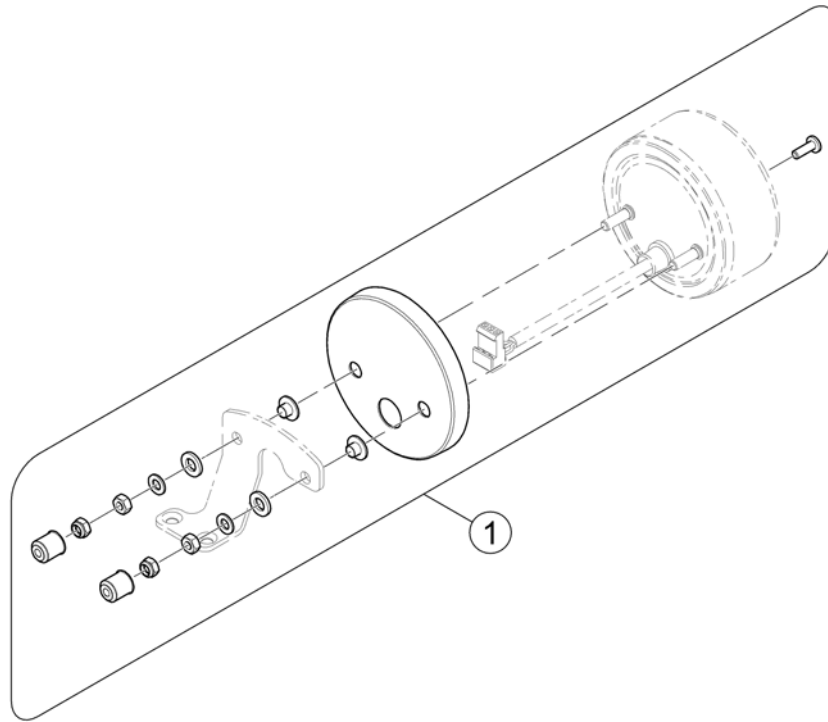

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581827	Rear Light Protector Exchange Kit		1

Lightsystem LED Front


1584022.iso

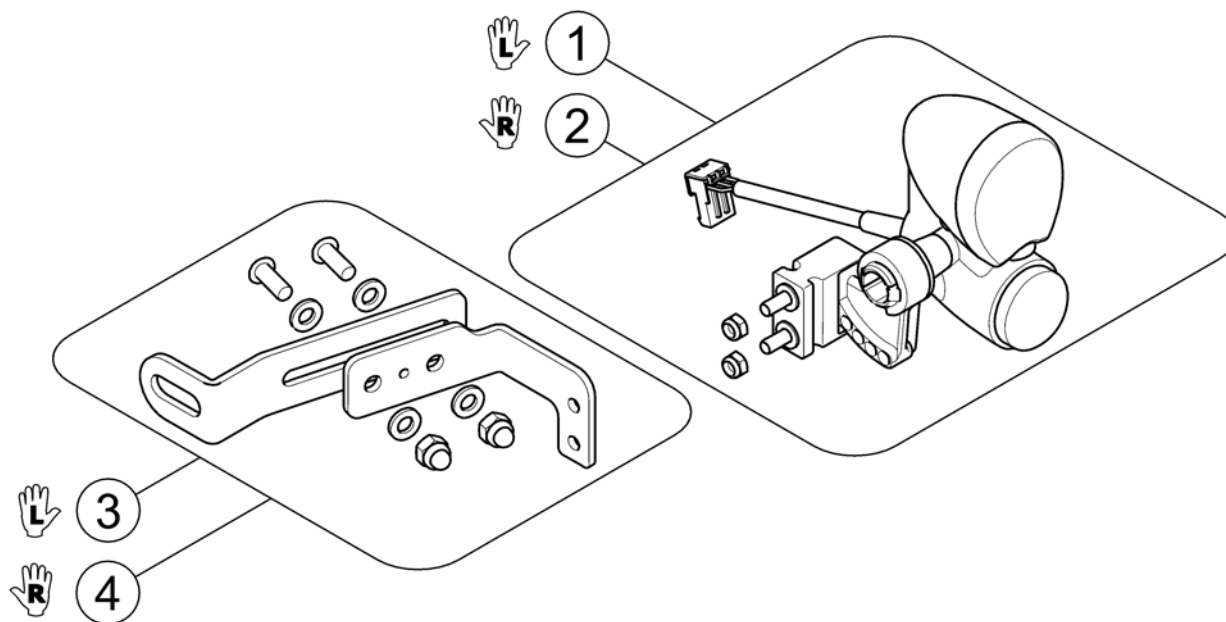

Pos.	Part number	Description	Valid from → until	Qty
1	SP1536030	Front light LED, LH		1
2	SP1576193	Front Light LED RH		1
3	SP1583093	Light Bracket LED, front LH		1
4	SP1583092	Light Bracket LED, front RH		1


Reflective Sign

SP1430410.iss

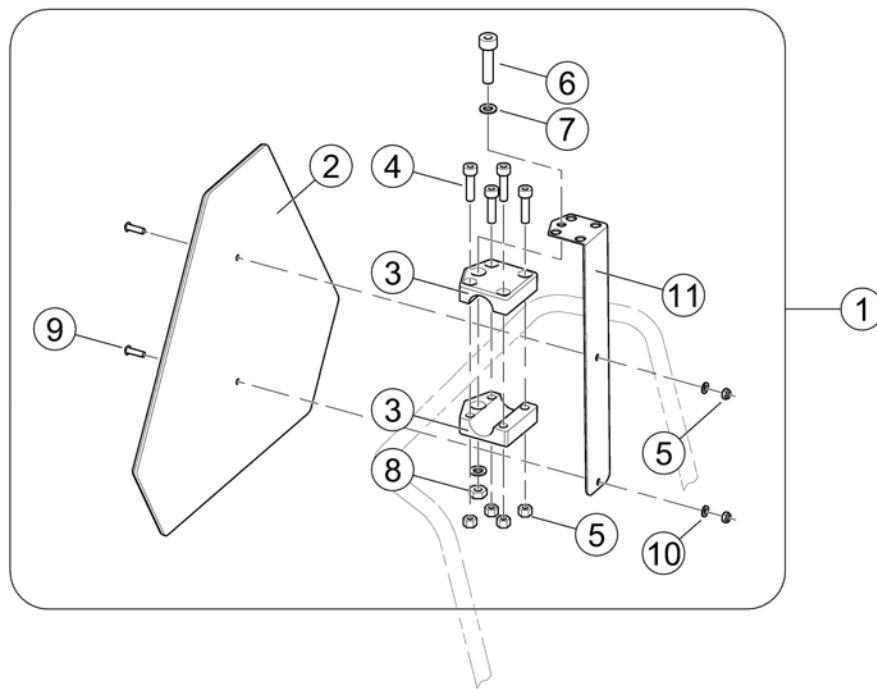

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430410	Reflective Sign ECE-R 69, complete		1
2	1430834	Rear Marking Sign ECE-R 69		1
3	Z12008A.P40	Gripping clip with hexagon hole, RAL 9005 black		2
4	D50004	Cylinder Bolt M6x30-8.8 bk - DIN 912		4
5	F20109	Hexagon Nut M6 bk - DIN 985		6
6	D00102	Hexagon Bolt M8x45-8.8 bk - DIN 931		1
7	G00107	Washer Ø8,4 bk - DIN 125		2
8	F00002	Hexagon Nut M8 bk - DIN 985		1
9	D90187	Fillister Head Screw M6x20-10.9 bk - DIN 7380		2
10	G00115	Washer Ø6,4 bk - DIN 125		2
11	1430418-9005	Mounting Bracket for Reflective Sign - RAL 9005 black		1
12	SP1580671	Bracket, Reflective Sign, cpl.		1


Lap Belt

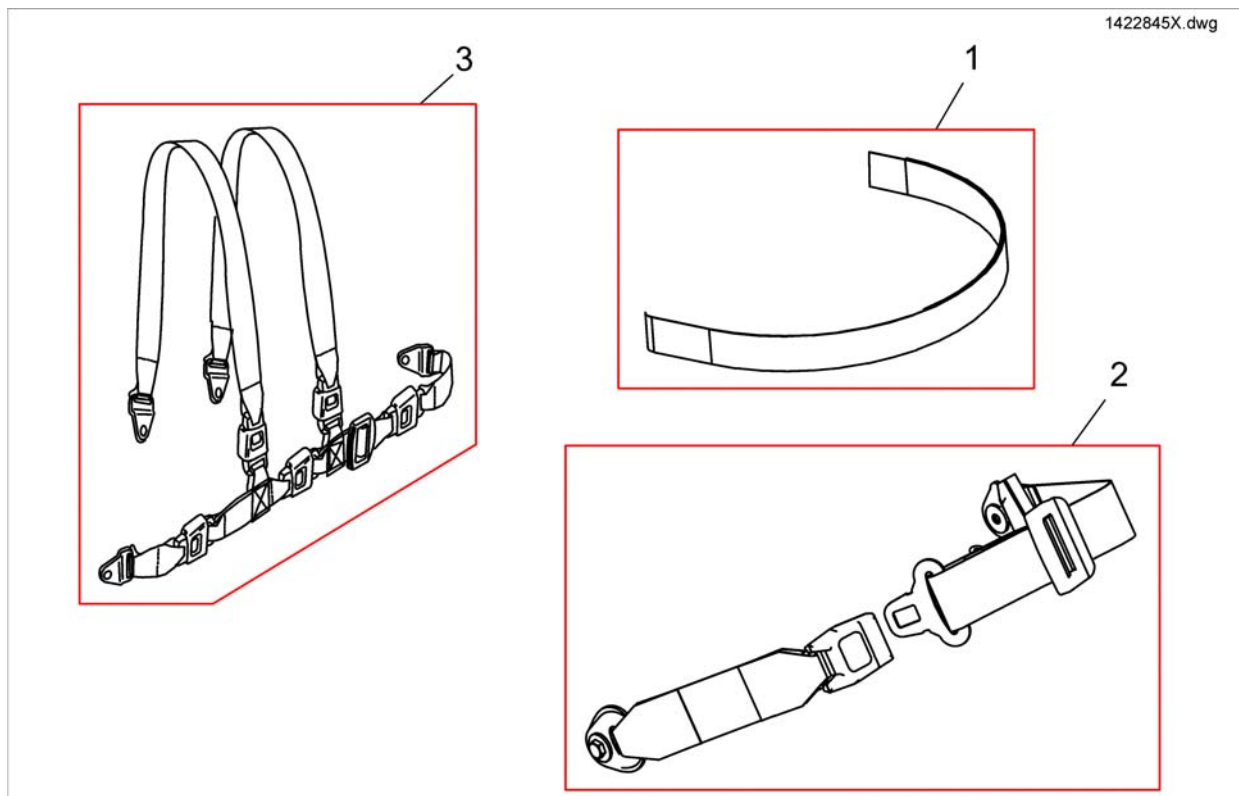

Pos.	Part number	Description	Valid from → until	Qty
1	1422870	POSITIONING STRAP WITH VELCRO CPL. FOR FLEX/CONTOUR-SEAT		1
1	1422869	Positioning Strap With Velcro cpl. for Junior-Seat		1
2	1422845	Lap Belt, cpl.		1
2	1422846	POSITIONING STRAP WITH SNAP CPL. FOR FLEX/CONTOUR		1
3	SP1422848	Holding Strap For Standard And Contour Seat H-Form cpl.		1
3	SP1422847	HOLDING STRAP FOR FLEX SEAT H-FORM CPL.		1


Lap Belts (Modulite Seat)

1571836 iso

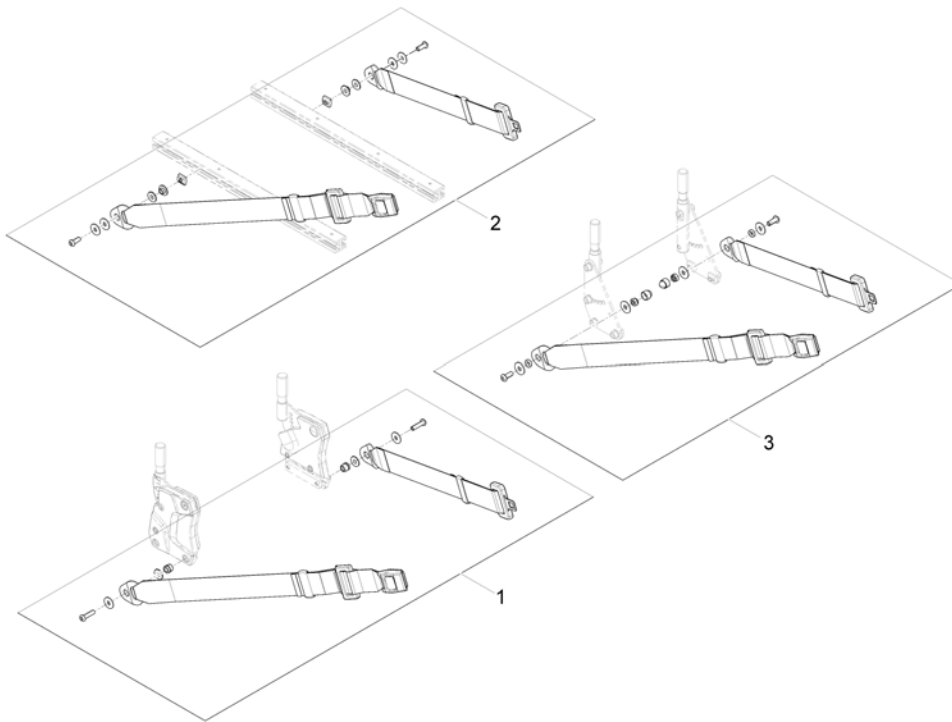


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570381	Lap Belt (fix mounting on Backrest Support), cpl.		1
2	SP1571183	Lap Belt (flexible mounting in side rails), cpl.		1
3	1422845	Lap Belt, cpl.		1

H-Belts (Modulite Seat)

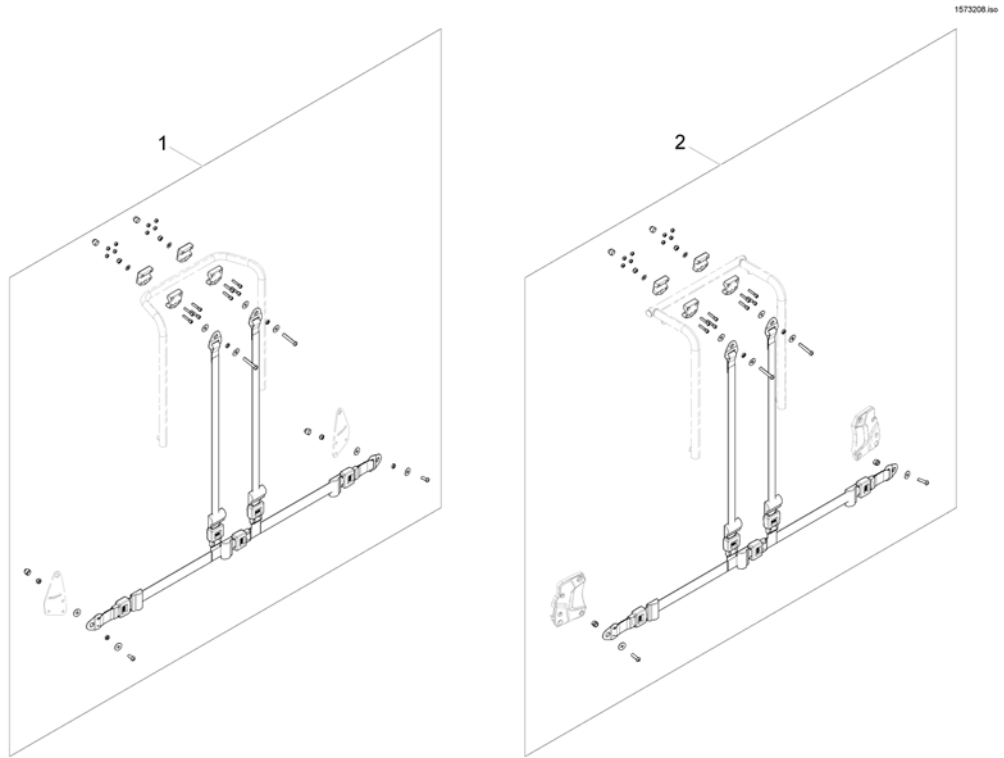

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570385	H-Belt for Backrest Frame		1
2	SP1570380	H-Belt for Back Posts		1


Rearview Mirror

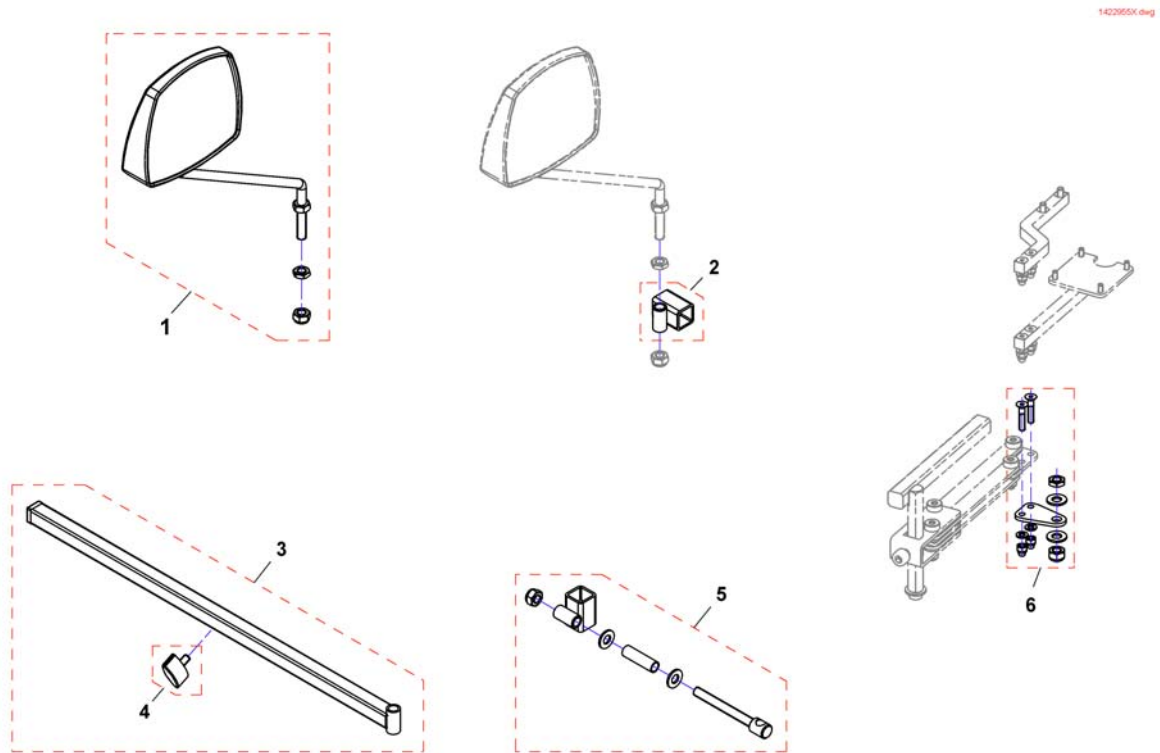

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422955	Rear Mirror		1
2	F276005.00	Mirror bracket		1
3	SP1423435	Rear Mirror Support Rail cpl.		1
4	L31001	Wing ScREW M6X10		1
5	SP1422957	Mirror bracket		1
6	SP1527151	Adapter to mount mirror and remote at the same side		1


Backrest Extension Flex

1573746.jpg

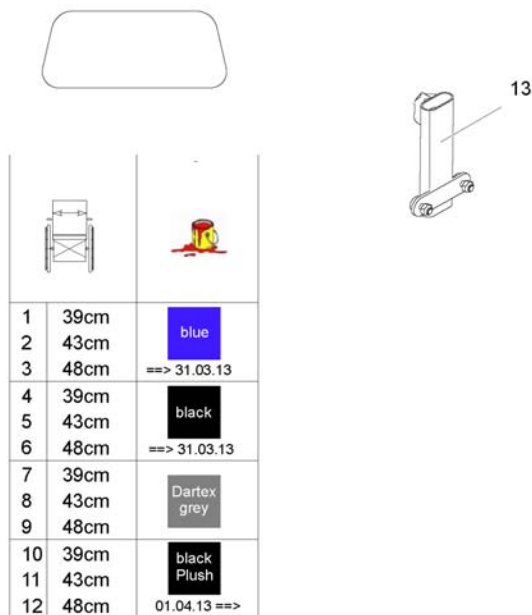

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422918	Backrest Extension for Flex Seat, SW39cm, blue	→ 2013-03-31	1
2	SP1422919	Backrest Extension for Flex Seat, SW43cm, blue	→ 2013-03-31	1
3	SP1422920	Backrest Extension for Flex Seat, SW48cm, blue	→ 2013-03-31	1
4	1436567	Backrest Extension for Flex Seat, SW39cm, black	→ 2013-03-31	1
5	1436568	Backrest Extension for Flex Seat, SW43cm, black	→ 2013-03-31	1
6	1440283	Backrest Extension for Flex Seat, SW48cm, black	→ 2013-03-31	1
7	5253035-TR23	Backrest Extension for Flex Seat, SW39cm, Dartex		1
8	5253036-TR23	Backrest Extension for Flex Seat, SW43cm, Dartex		1
9	5253037-TR23	Backrest Extension for Flex Seat, SW48cm, Dartex		1
10	SP1436567-TR35	Backrest Extension for Flex Seat, SW39cm, black Plush	2013-04-01 →	1
11	SP1436568-TR35	Backrest Extension for Flex Seat, SW43cm, black Plush	2013-04-01 →	1
12	SP1440283-TR35	Backrest Extension for Flex Seat, SW48cm, black Plush	2013-04-01 →	1
13	SP1422921	Fixation for Backrest Extension		1

Abduction Wedge

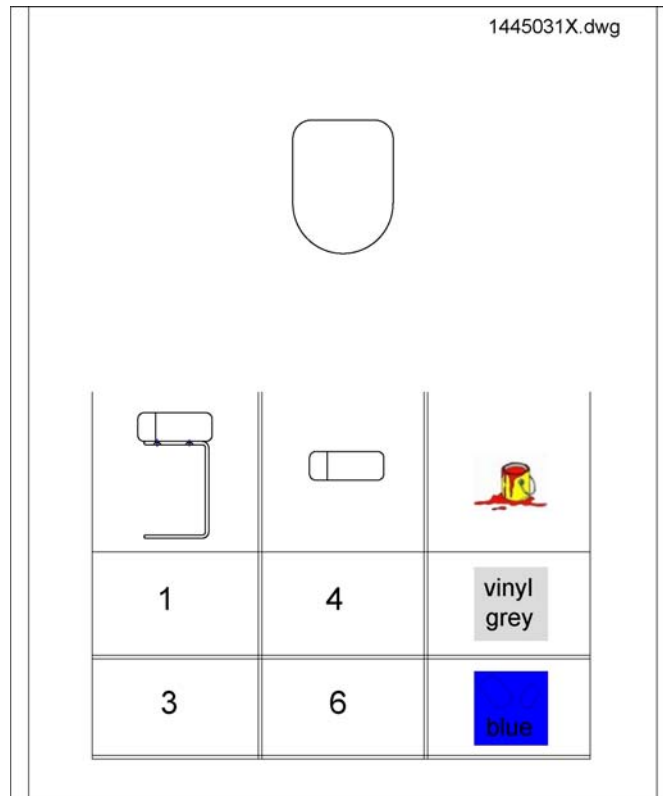


Pos.	Part number	Description	Valid from → until	Qty
1	SP1445033	Abduction Wedge Junior, grey Vinyl, with Bracket		1
3	SP1445032	Abduction Wedge Junior, Blue Dot, with Bracket		1
4	SP5313257	Abduction Wedge Junior, grey Vinyl, wo Bracket		1
6	SP5313256	Abduction Wedge Junior, Blue Dot, wo Bracket		1

Lateral Support - Flex 2 Seat

Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral Support (Dartex) [1446]		1
2	-	Lateral Support, extended arm (Dartex) [1445]		1

Lateral Support (Dartex)


1526377.000

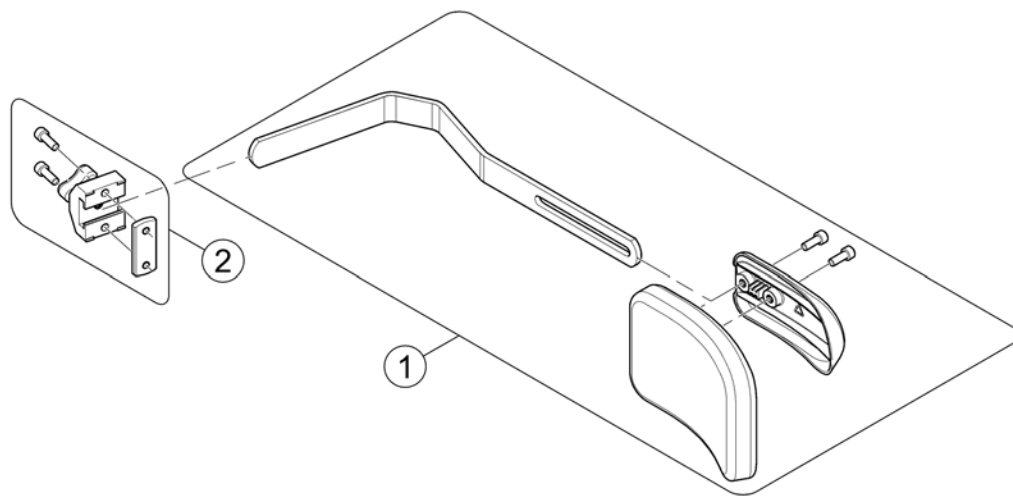

Pos.	Part number	Description	Valid from → until	Qty
1	SP1561841	Trunk support, cpl.		1
2	5253170	Lateral Support Bracket		1

Lateral Support, extended arm (Dartex)

1562674.iss

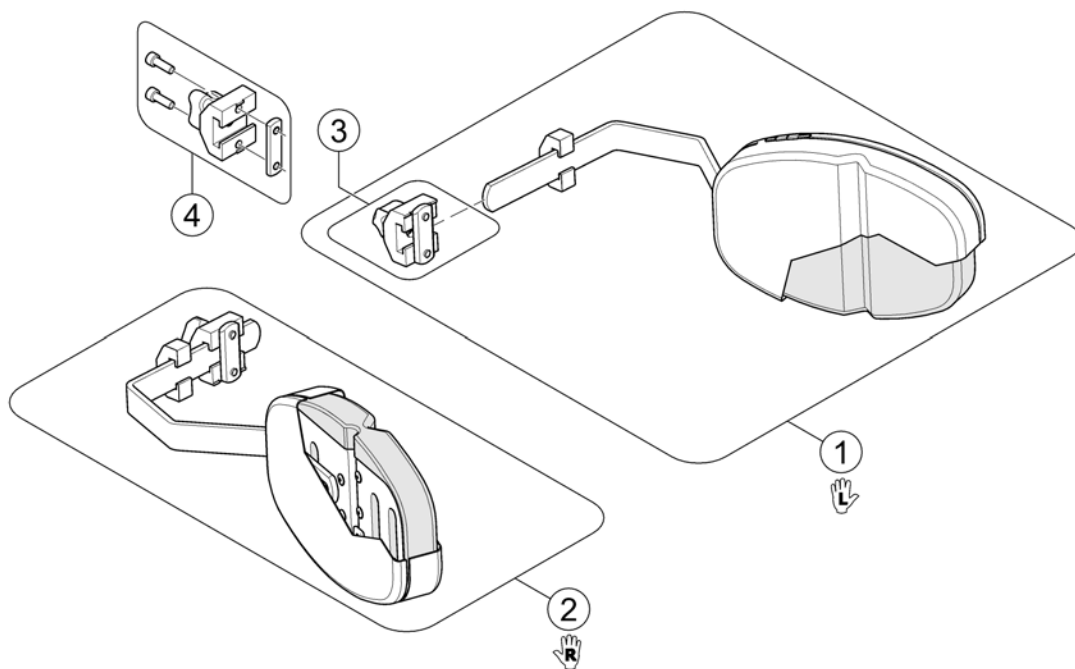


Pos.	Part number	Description	Valid from → until	Qty
1	1448692	Lateral Support, multi adjustable - RH <i>Dartex grey</i>		1
2	1448693	Lateral Support, multi adjustable - RH <i>Dartex grey</i>		1
3	5253170	Lateral Support Bracket <i>Only for Flex/Flex2 Seat!</i>		1
4	1543574	Locking trunk support <i>Only for Modulite Seat!</i>		1

Lateral Support - Modulite Seat

Pos.	Part number	Description	Valid from → until	Qty
1	-	Lateral Support (Dartex) [1446]		1
2	-	Lateral Support, extended arm (Dartex) [1445]		1

Lateral Support (Dartex)


158860 iso

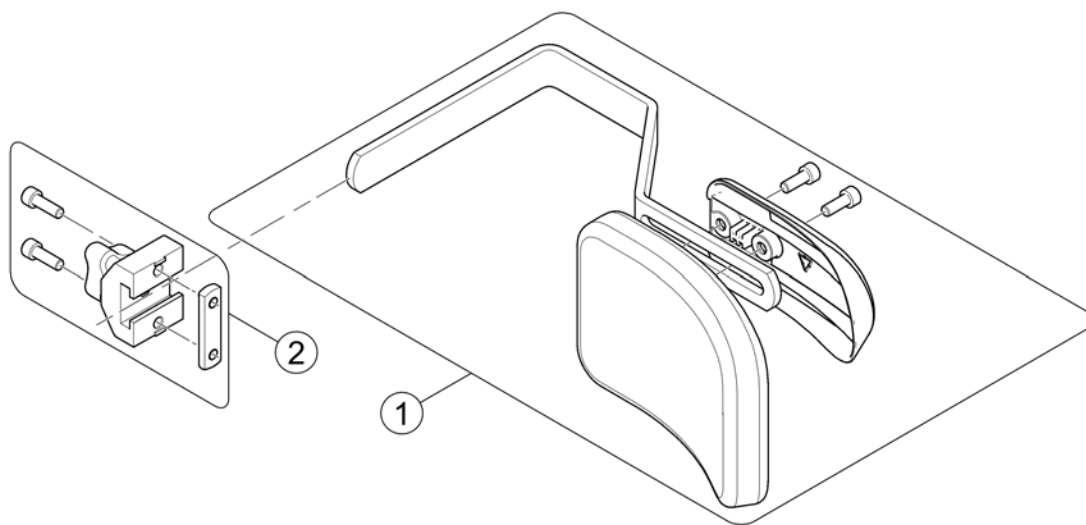

Pos.	Part number	Description	Valid from → until	Qty
-		Design change on 01. July 2015 - we suggest to change older Supports on both sides.		0
1	SP1588314	Lateral Support, cpl.		1
2	1543574	Locking trunk support		1

Lateral Support - Junior Seat

5313271X.dwg

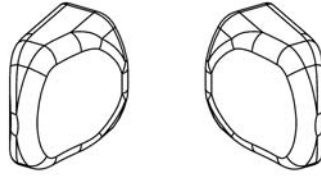


RH	LH	
1	2	blue dot
3	4	vinyl grey

Pos.	Part number	Description	Valid from → until	Qty
1	SP5313271	Trunk Support RH for Junior, blue Dot		1
2	SP5313275	Trunk Support LH for Junior, blue Dot		1
3	5313272	TRUNK SUPPORT VINYL BLACK RH CPL.		1
4	5313276	TRUNK SUPPORT VINYL BLACK LH CPL.		1


Head- & Neckrests

1422865X.dwg

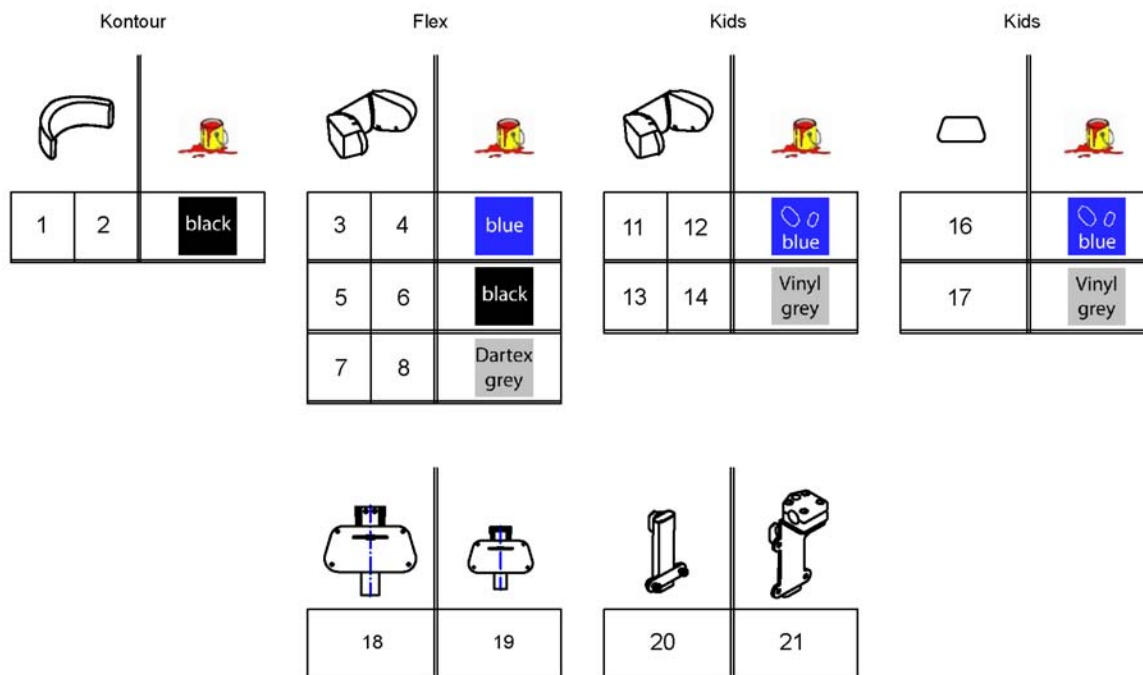

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422925	Headrest, standard/contour seat, black, complete		1
2	SP1422865	Headrest cushion contour black, without bracket		1
3	-	No longer available.		0
4	-	No longer available.		0
5	SP1440285-TR35	Headrest Flex, Plush black, cpl.		1
6	SP1585039	Headrest Cushion Flex, Plush black (chenille TR35), w/o bracket		1
7	SP1428092	Headrest Flex, Dartex grey, complete		1
8	SP5252229	Headrest cushion Flex Dartex grey, without Bracket		1
11	SP5313307	Headrest cushion Junior, blue Dot, complete		1
12	SP5313261	Headrest cushion Junior, blue Dot, without bracket		1
13	SP5313308	Headrest Cushion Junior, Vinyl grey, complete		1
14	SP5313262	Headrest Cushion Junior, Vinyl grey, without bracket		1
16	SP5313311	Neckrest Cushion Junior, Blue Dot		1
17	SP5313312	Neckrest Cushion Junior, grey Vinyl		1
18	SP1419636	Bracket for Neckrest		1
19	SP1419637	Neckrest Holder Typ3 short		1
20	SP1422547	Headrest Bracket, Contour-/Flex Seat, complete		1
21	SP1422914	Head-/Neckrest Bracket, cpl.		1

Push Handle

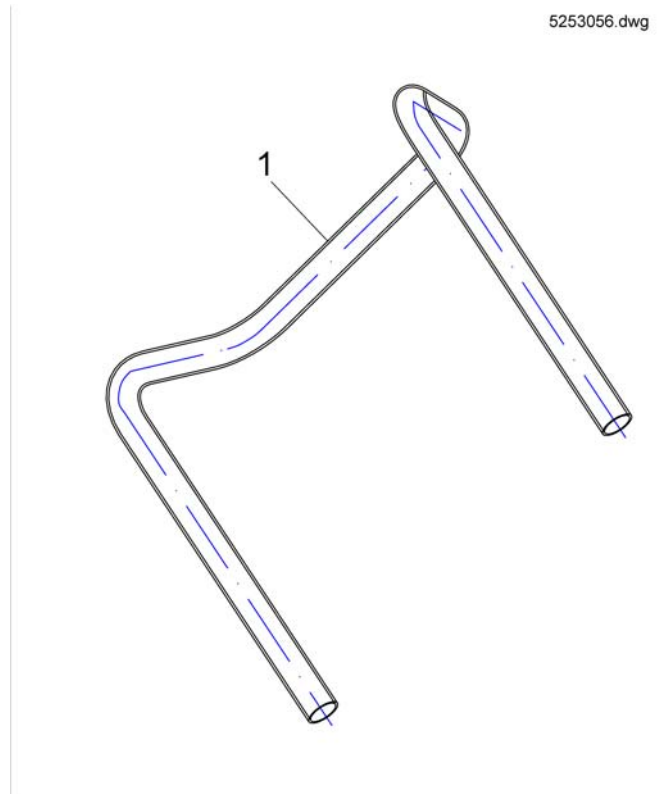


Pos.	Part number	Description	Valid from → until	Qty
1	SP1425839	Push Handle Flex SW 43/48cm		1
1	SP1425840	Push Handle Flex SW 39cm		1
2	SP1524672	Push Handle (single one)		1

Caneholder

Pos.	Part number	Description	Valid from → until	Qty
-		Caneholder for Standard Seat		1
-		Caneholder for Contour/Flex Seat		1
-		Cane holder (Modulite: backrest frame)		1
-		Cane holder (Modulite: back posts man recliner)		1
-		Cane holder (Modulite: back posts pow recliner)		1

Caneholder for Standard Seat


1567105.iso

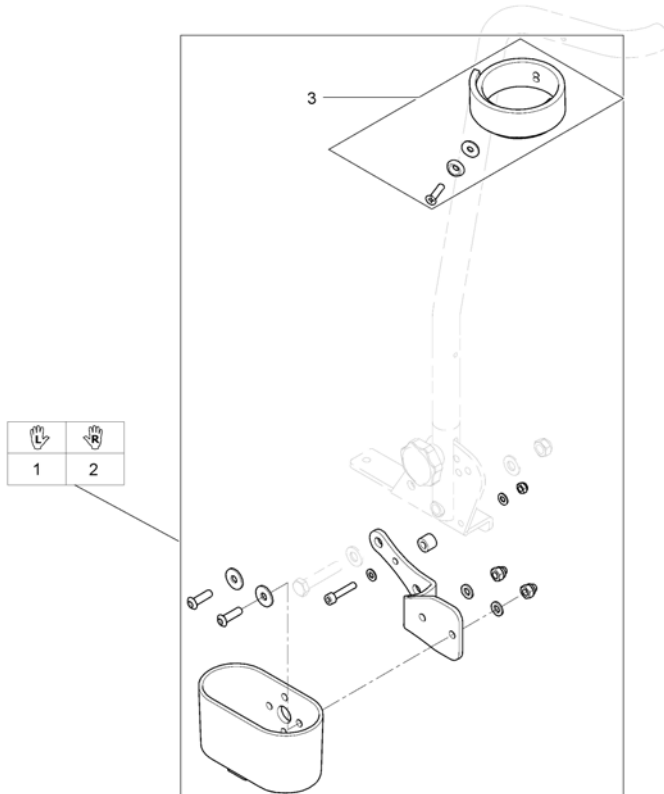

Pos.	Part number	Description	Valid from → until	Qty
1	1442701	CANE HOLDER FOR STANDARD-SEAT, CPL. Tornado/Typhoon		1
2	SP1534877	Cane Holder for Standard Seat, cpl.		1
3	SP5009551	Belt for Caneholder		1


Caneholder for Contour/Flex Seat

1567254.iso

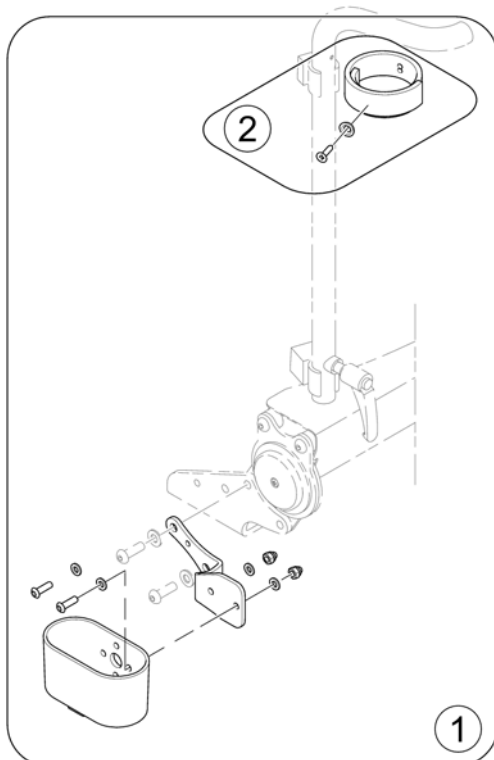


Pos.	Part number	Description	Valid from → until	Qty
1	1442702	Cane Holder For Contour/Flex-Seat, cpl.		1
2	SP5009551	Belt for Caneholder		1

Invacare® TDX-SP, TDX-SP nB, TDX Sprint | Accessories
Cane holder (Modulite: backrest frame)

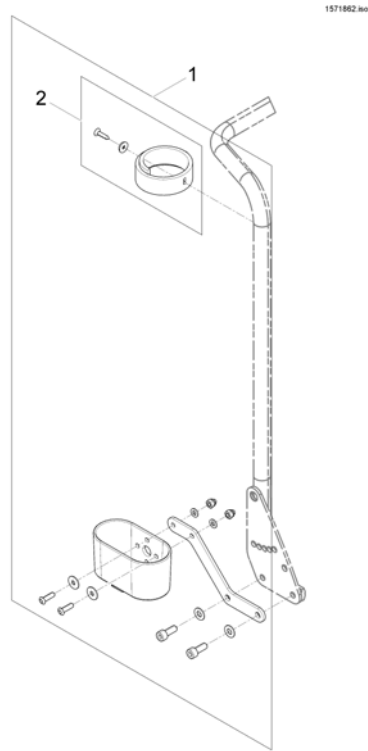


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570396	Cane holder, standard backrest		1
2	SP1420493	Belt for caneholder		1

Cane holder (Modulite: back posts man recliner)

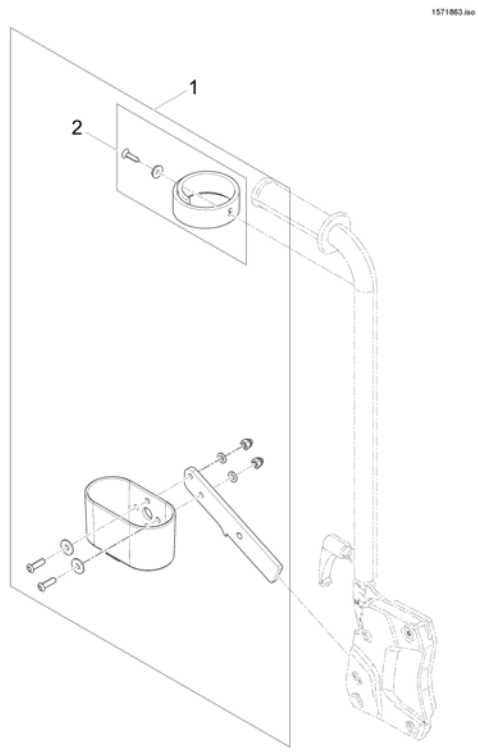


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570443	Cane holder, manual backrest		1
2	SP1420493	Belt for caneholder		1

Cane holder (Modulite: back posts pow recliner)

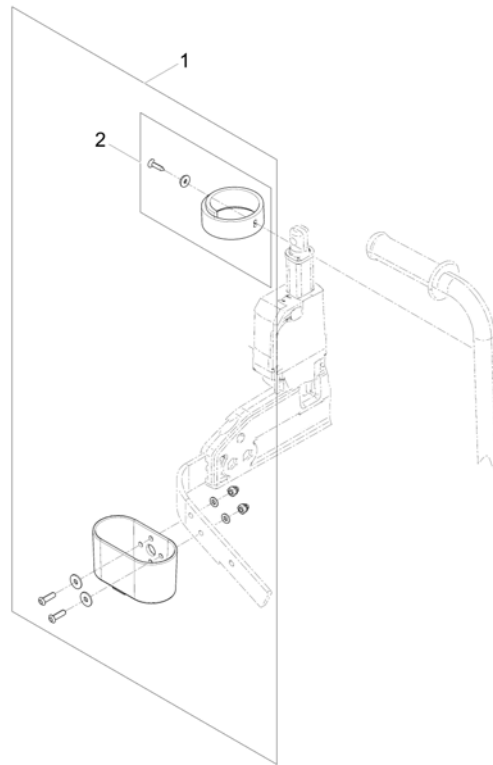

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570382	Cane holder, powered backrest		1
2	SP1420493	Belt for caneholder		1

Trays

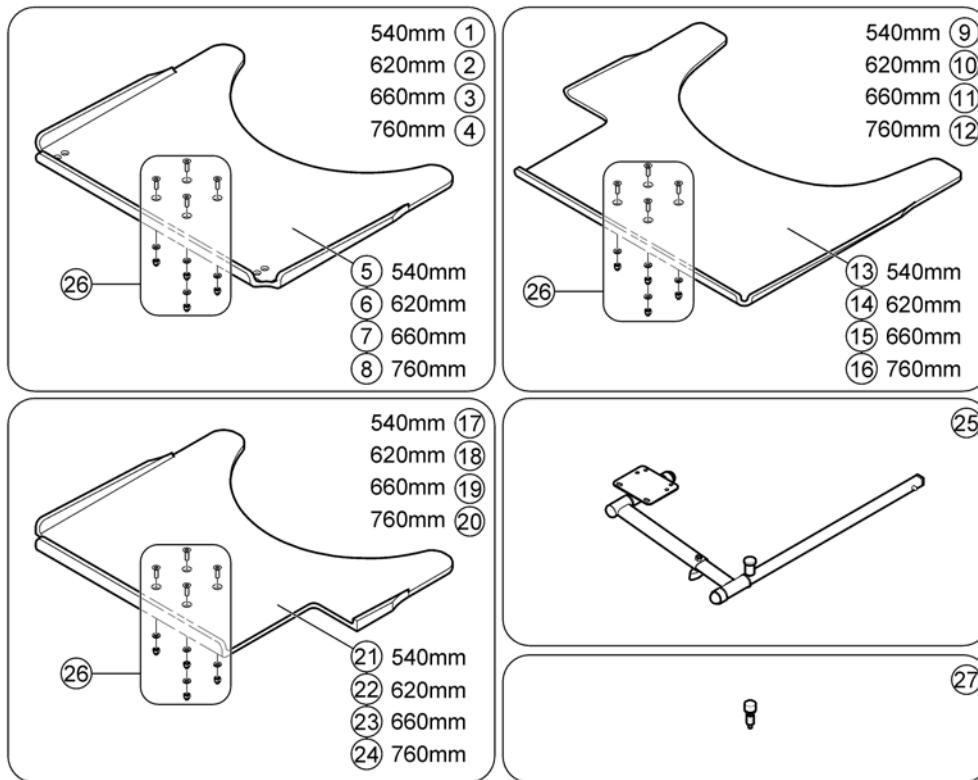


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548198	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, no Cutout, cpl. with holder		1
2	SP1548201	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, no Cutout, cpl. with holder		1
3	SP1422954	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, no Cutout, cpl. with holder		1
4	SP1584644	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, no Cutout, cpl. with holder		1
5	SP1535050	Tray table, 540mm, no Cutout		1
6	SP1535010	Tray table, 620mm, no Cutout		1
7	SP1535011	Tray table, 660mm, no Cutout		1
8	SP1586049	Tray table, 760mm, no Cutout		1
9	SP1548138	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout RH, cpl. with holder		1
10	SP1548199	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout RH, cpl. with holder		1
11	SP1422953	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout RH, cpl. with holder		1
12	SP1584663	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout RH, cpl. with holder		1
13	SP1535051	Tray table, 540mm, Cutout RH		1
14	SP1535012	Tray table, 620mm, Cutout RH		1
15	SP1535049	Tray table, 660mm, Cutout RH		1
16	SP1586048	Tray table, 760mm, Cutout RH		1
17	SP1548197	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout LH, cpl. with holder		1
18	SP1548200	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout LH, cpl. with holder		1
19	SP1422952	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout LH, cpl. with holder		1
20	SP1584662	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout LH, cpl. with holder		1
21	SP1535009	Tray table, 540mm, Cutout LH		1
22	SP1534974	Tray table, 620mm, Cutout LH		1

Trays

Pos.	Part number	Description	Valid from → until	Qty
23	SP1534975	Tray table, 660mm, Cutout LH		1
24	SP1586047	Tray table, 760mm, Cutout LH		1
25	SPZ177011	Tray Support Fixing Kit cpl.SW 380/505 mm		1
26	Z177025	Fastening-Kit Tray		1
27	SP1590090	Table Bumper high version		1

Hour Counter


1567259.iso

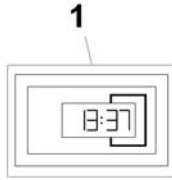

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530275	Hour counter		1

KLICKfix® bag


1562500 iso

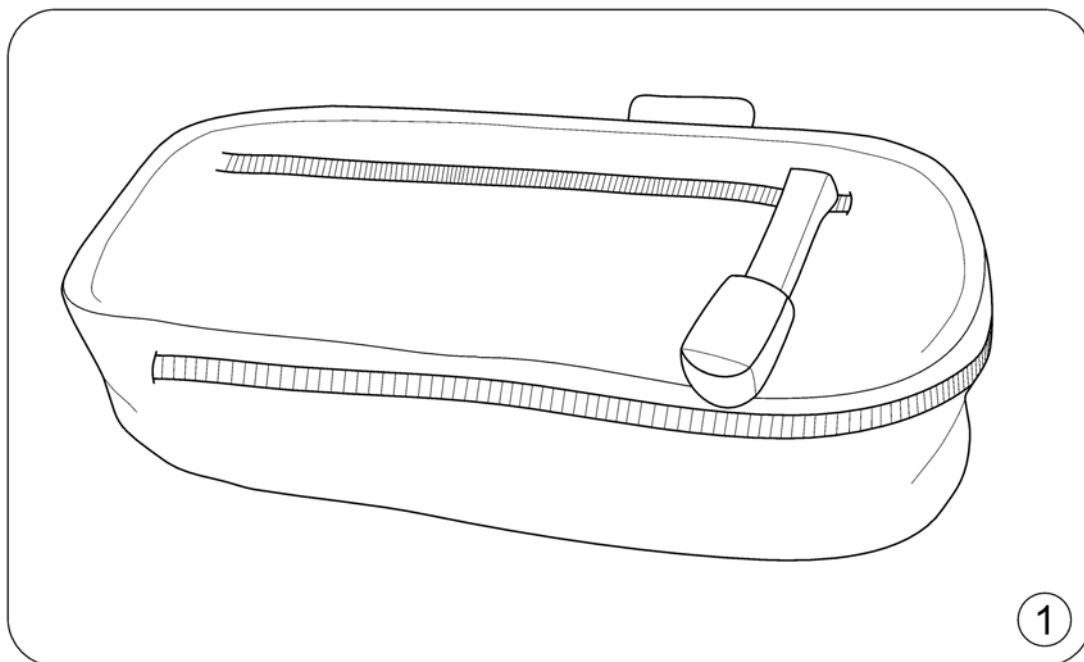

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557532	KLICKfix® bag		1

KLICKfix® bracket for Fixed Armrest


1562515.iss

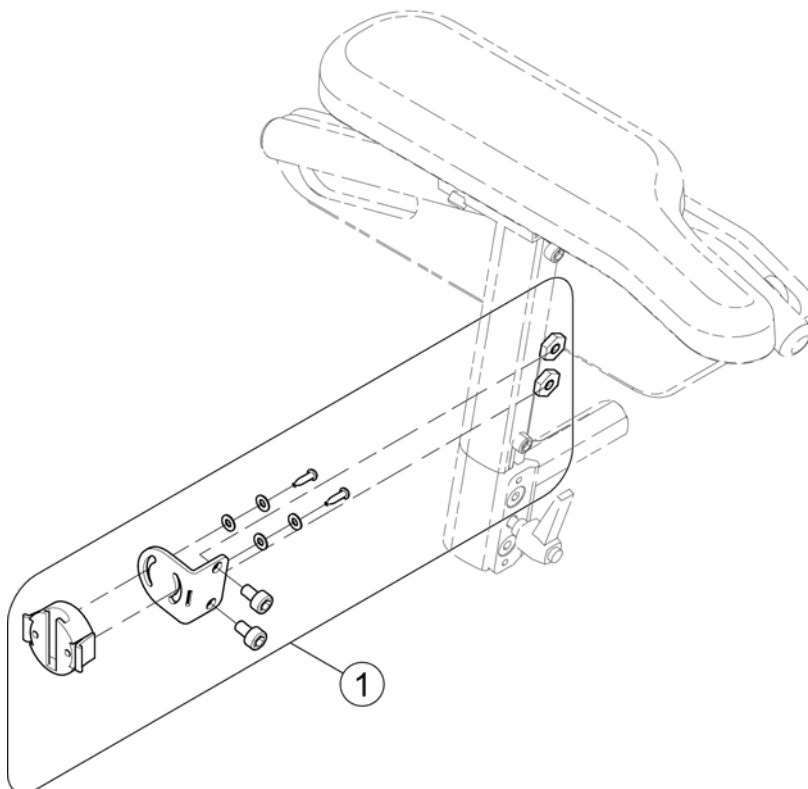

Pos.	Part number	Description	Valid from → until	Qty
1	SP1554320	KLICKfix bracket		1

KLICKfix® bracket for T-Armrest


1571871.iso

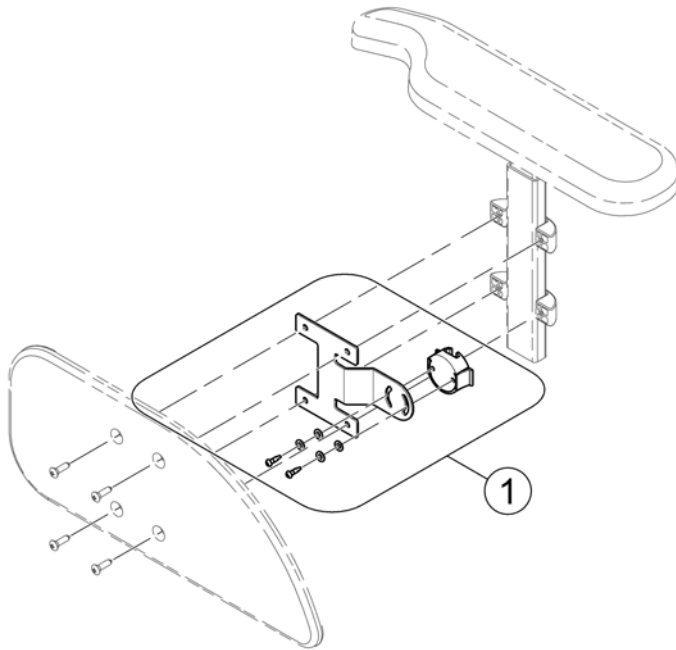

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571678	KLICKfix® bracket		1

KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite)


1580167 Rev

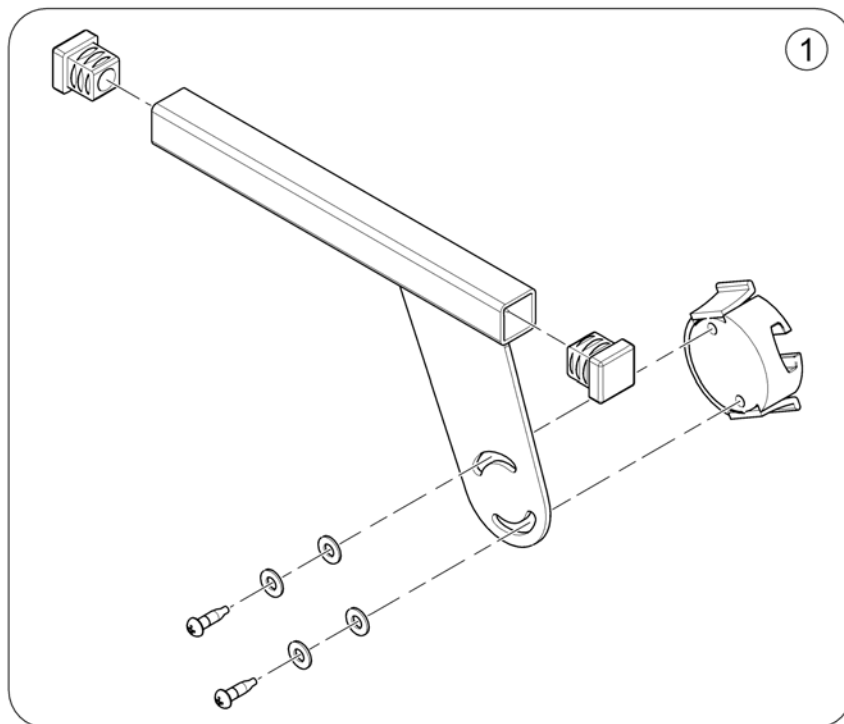

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584497	KlickFix Bracket		1

Bag


1567256 iso

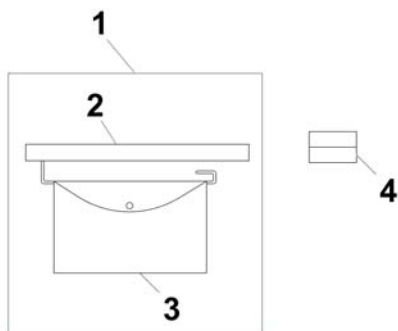

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430417	Bag for side panel, cpl. LH		1
2	SP1430855	Holder for armrest bag		1
3	SP5314967	Bag for armrest		1
4	F370032-9005	Holder for side part with mirror/brake RAL 9		1

Bumper/Sport Accessories


1537296X.ilo

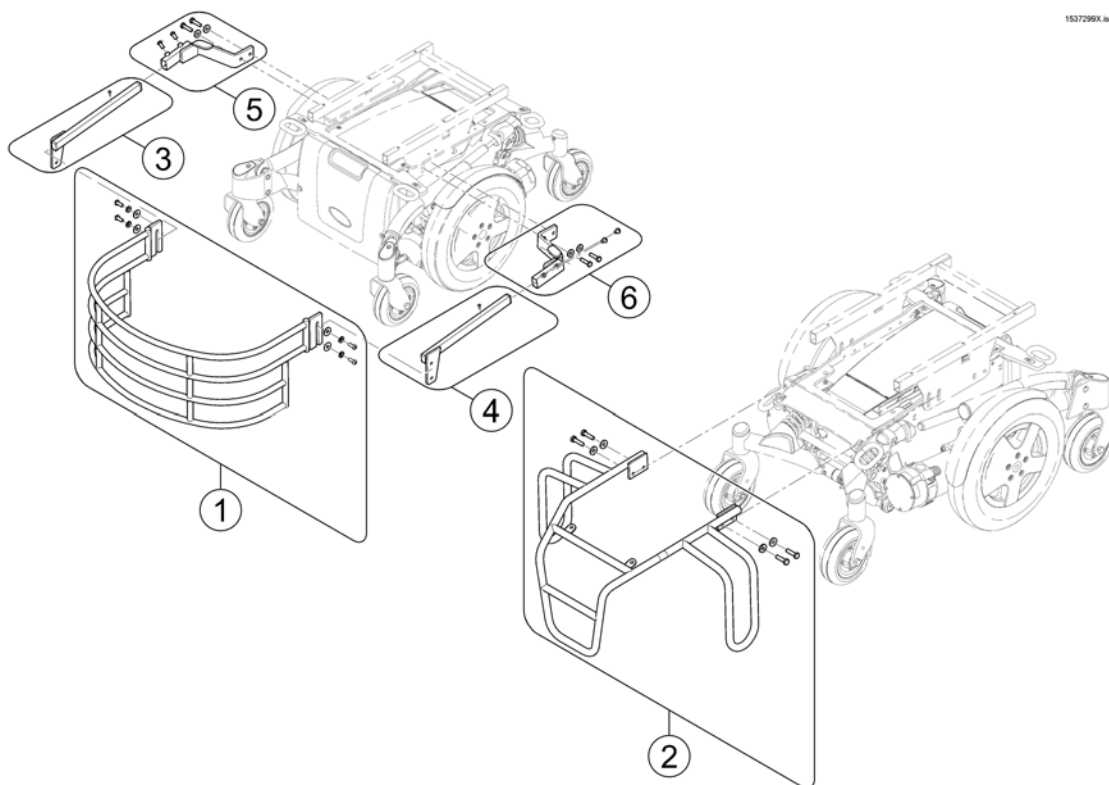

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504203	Front Bumper, rubberrised, cpl.		1
2	SP1537300	Rear bumper, cpl.		1
3	SP1537894	Holder frontbumper, RH		1
4	SP1537895	Holder frontbumper, LH		1
5	SP1537896	Frameholder, RH		1
6	SP1537897	Frameholder, LH		1

Invacare® TDX-SP, TDX-SP nB, TDX Sprint | Accessories
T-Nuts for Accessories Modulite Seat

157-4067 Rev

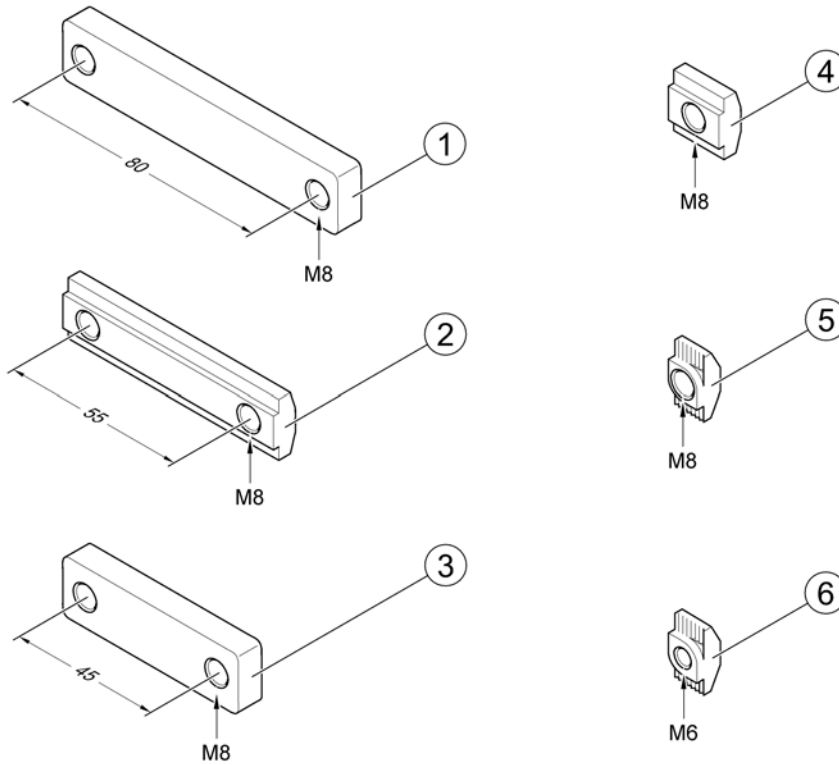

Pos.	Part number	Description	Valid from → until	Qty
1	SP1556401	T-Nut, hole distance 80mm, 2x M8		1
2	SP1572444	T-Nut, hole distance 55mm, 2x M8		1
3	SP1555168	T-Nut, hole distance 45mm, 2x M8		1
4	SP1557341	T-Nut, 1x M8		1
5	SP1573158	T-Nut, 1x M8		1
6	SP1573721	T-Nut, 1x M6		1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånå
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk