

# *Spare Parts Catalogue*

---

Version: 2015/05/18c EN



**Invacare®  
TDX-SP2 / TDX-SP2 NB**



**Yes, you can.®**

## **General Information**

*For any order of spare parts the following information is required:*

- Product type*
- Product name & model*
- Serial Number*
- Part-Number*
- Part-Description*

*Exploded view drawings and code numbers are not contractual and subject to modification, in relation to Invacare® continuous product improvement policy to meet end users needs.*

*The reproduction or use of this work, or of any substantial part of this work, in any form or by any electronic, mechanical or other means, including photocopying, scanning and recording, and in any information storage and retrieval system, including electronic transmission, is forbidden without the prior written permission of Invacare® International Sarl.*



Yes, you can.®

## TABLE OF CONTENTS

---

### Information & User Manuals

User Manuals TDX-SP2 .....	11
User Manuals Shark2 .....	12
User Manuals REM A & REM B .....	13
User Manuals REM24SD .....	14
User Manuals REM550 .....	15
User Manuals G90 .....	16
User Manuals G91 .....	17
User Manuals REM34 & REM41 Tray/Centre Control .....	18
User Manuals IDC Dual Control .....	19
User Manuals ACS Handprogrammer .....	20
User Manual Proximity Head Remote DSL-105 .....	21
User Manuals Headcontrol .....	22
User Manuals iPortal .....	23

### Chassis

Chassis .....	24
Chassis Wide Base .....	25
Battery Box Hardware .....	26
Walking Beam .....	27
Rear Pivot Wide Base .....	28
Shrouds Wide Base .....	29
Rear Shrouds - Wide Base .....	30
Bezel Shroud .....	31
Front Shroud - Wide Base .....	32
Fender .....	33
Rear Caps .....	34
Low Rider Option .....	35
Chassis Narrow Base .....	36
Battery Box Hardware .....	26
Walking Beam .....	27
Rear Pivot Narrow Base .....	37
Shrouds Narrow Base .....	38
Rear Shrouds Narrow Base .....	39
Bezel Shroud .....	31
Front Shroud Narrow Base .....	40
Fender .....	33
Rear Caps .....	34

### Seat

Seat .....	41
Modulite Seat .....	42
man. settable Tilt .....	43
powered simple Tilt 20° .....	44
powered Tilt 0°-30° .....	45
Lifter .....	46
One Piece Seat Plate .....	47
Telescopic Seat Frame and Plate .....	48
Telescopic Seat Frame for man. /pow. 20° Tilt .....	49
Telescopic Seat Frame for Tilt 30° / Lifter .....	50



Yes, you can.®

## TABLE OF CONTENTS

---

Armrest Bracket .....	51
Legrest Support .....	52
Seat Plate .....	53
Telescopic Seat Frame and Sling Seat .....	54
Telescopic Seat Frame for man. / pow. 20° Tilt .....	55
Telescopic Seat Frame for Tilt 30° / Lifter .....	56
Armrest Bracket .....	51
Legrest Support .....	57
Sling Seat .....	58
Telescopic Seat Frame and Universal Adapter .....	59
Telescopic Seat Frame for man. /pow. 20° Tilt .....	49
Telescopic Seat Frame for Tilt 30° / Lifter .....	50
Armrest Bracket .....	51
Legrest Support .....	52
Adapter Kit for Universal Seat .....	60
Seat Cushion .....	61
Max Seat .....	64
man. settable Tilt .....	65
powered simple Tilt .....	66
Seat Frame and Plate .....	67
Seat Cushion .....	68

### Backrest

Backrest .....	69
Backrest for Modulite Seat .....	70
Support for Backrest Frame .....	71
Backrest Frame .....	72
Sling Back, tension adjustable .....	73
Backrest support .....	74
Backposts .....	75
Connection tube .....	76
Back Recliner / Stabilization bracket .....	77
Backrest Cushion .....	78
Flex3 adjustable Backrest Plate .....	80
Backrest for Max Seat .....	81
Manual backrest adjustment .....	82
Powered backrest adjustment .....	83
Seat depth adapter .....	84
Backrest .....	85

### Legrests

Legrests .....	86
Legrests - Wide Base .....	87
Modulite Seat .....	88
Legrest Support 5cm, Vari series .....	89
Hangers Vari A LH/RH 04.2009 => .....	90
Hangers Vari F LH/RH .....	91
Hangers ADM LH/RH manual .....	92
Hangers ADE/ADP LH/RH electric 07.2005 => .....	93
Footrests Vari, Flip-Up - 01.08.10 => .....	94
Footrests Vari, Flip-Up & angle adjustable - 01.08.10 => .....	95



Yes, you can.®

## TABLE OF CONTENTS

---

Footrests Vari, angle & depth adjustable .....	96
Footboard Vari, pre-assembled .....	97
Support Centr. Legrest LNX man. - Modulite Seat .....	98
Support Centr. Legrest LNX el. - Modulite Seat .....	99
Centr. Legrest man. ....	100
Centr. Legrest LNX el. ....	101
Max Seat .....	102
Legrest Support 5cm, Vari series .....	89
Hangers Vari A LH/RH 04.2009 => .....	90
Hangers Vari F LH/RH .....	91
Hangers ADM LH/RH manual .....	92
Hangers ADE/ADP LH/RH electric 07.2005 => .....	93
Footrests Vari, Flip-Up - 01.08.10 => .....	94
Footrests Vari, Flip-Up & angle adjustable - 01.08.10 => .....	95
Footrests Vari, angle & depth adjustable .....	96
Footboard Vari, pre-assembled .....	97
Support Centr. Legrest man. LNX - Max Seat .....	103
Support Centr. Legrest LNX el. - Max Seat .....	104
Centr. Legrest man. ....	100
Centr. Legrest LNX el. ....	101
Fixed footboard .....	105
Calf plates .....	106
Legrests - Narrow Base .....	107
Legrest Support 5cm, Vari series .....	89
Hangers Vari A LH/RH 04.2009 => .....	90
Hangers Vari F LH/RH .....	91
Hangers ADM LH/RH manual .....	92
Hangers ADE/ADP LH/RH electric 07.2005 => .....	93
Footrests Vari, Flip-Up - 01.08.10 => .....	94
Footrests Vari, Flip-Up & angle adjustable - 01.08.10 => .....	95
Footrests Vari, angle & depth adjustable .....	96
Support Centr. Legrest LNX man. - Modulite Seat .....	98
Support Centr. Legrest LNX el. - Modulite Seat .....	99
Centr. Legrest man. ....	100
Centr. Legrest LNX el. ....	101

### Armrests

Armrests .....	108
Armrests for Modulite Seat .....	109
T-Armrest .....	110
Padded Armrest .....	111
Inrigged Armrest .....	112
Flip-Up Armrest .....	113
Following Armrest .....	114
Hip Support .....	115
Armrests for Max Seat .....	116

### Castors

Castors .....	117
---------------	-----

### Drive Wheels



Yes, you can.®

## TABLE OF CONTENTS

---

Drive Wheels 14" .....	118
Wheel Hub 5-hole .....	119

### Propulsion

Propulsion SSD - Details .....	120
Propulsion SSD - Wide Base .....	123
Propulsion SSD - Narrow Base .....	124
Wheel Hub 5-hole .....	119

### Electronic

Electronic .....	125
Electronic for Wide Base .....	126
ACS Electronic Modules .....	127
Power Modul - ACS .....	128
Bracket for Power Modul Wide Base .....	129
ACT for ACS 2 .....	130
Bracket for ACT Wide Base .....	131
Light Board .....	132
G-Trac-sensor .....	133
ACS Remote & Brackets .....	134
Remote REM 24 SD with Brackets .....	135
Remote REM 24SD .....	136
Remote Bracket Standard .....	137
Remote Bracket Swing away & height adjust. ....	138
Remote Bracket Attendant .....	139
Remote REM A & B with Brackets .....	140
Remote Bracket Standard .....	137
Remote Bracket Swing away & height adjust. ....	141
Remote Bracket Attendant .....	139
Remote REM 550 with Brackets .....	142
Remote Bracket Standard .....	137
Remote Bracket Swing away & height adjust. ....	138
Remote Bracket Attendant .....	139
Remote REM 34 with Brackets .....	143
Remote Bracket Standard .....	144
Remote Bracket Swing away & height adjust. ....	145
Remote G90A with Brackets .....	146
Remote Bracket Standard .....	147
Remote Bracket Swing away & height adjust. ....	148
Remote Bracket Attendant .....	149
Remote G91S with Brackets .....	150
Remote Bracket Standard .....	151
Centre control ACS REM41 .....	152
Remote Small Chin Control with Brackets .....	153
Small Chincontrol .....	154
Framework for chincontrol .....	155
Remote Dual Control with Brackets .....	156
Remote Bracket Dual Control for Max Seat .....	157
Remote Bracket Dual Control for Modulite Seat .....	158
Remote IDC Dual Control with Brackets .....	159
ACS Cables & Accessories .....	160



Yes, you can.®

## TABLE OF CONTENTS

---

Bus-Cables ACS .....	161
Light Cable ACS .....	162
T-Adapter - Modulite & Max Seat .....	163
Programming Kits .....	164
Magnetkey for Remotes .....	165
Electronic for Narrow Base .....	166
ACS Electronic .....	167
ACS Electronic Modules .....	168
Power Modul - ACS .....	128
Bracket for Power Modul Narrow Base .....	169
ACT for ACS 2 .....	130
Bracket for ACT Narrow Base .....	170
Light Board .....	171
G-Trac-sensor .....	133
ACS Remote & Brackets .....	172
Remote REM 24 SD with Brackets .....	135
Remote REM 24SD .....	136
Remote Bracket Standard .....	137
Remote Bracket Swing away & height adjust. ....	138
Remote Bracket Attendant .....	139
Remote REM A & B with Brackets .....	140
Remote Bracket Standard .....	137
Remote Bracket Swing away & height adjust. ....	141
Remote Bracket Attendant .....	139
Remote REM 550 with Brackets .....	142
Remote Bracket Standard .....	137
Remote Bracket Swing away & height adjust. ....	138
Remote Bracket Attendant .....	139
Remote REM 34 with Brackets .....	143
Remote Bracket Standard .....	144
Remote Bracket Swing away & height adjust. ....	145
Remote G90A with Brackets .....	146
Remote Bracket Standard .....	147
Remote Bracket Swing away & height adjust. ....	148
Remote Bracket Attendant .....	149
Remote G91S with Brackets .....	150
Remote Bracket Standard .....	151
Centre control ACS REM41 .....	152
Remote Small Chin Control with Brackets .....	153
Small Chincontrol .....	154
Framework for chincontrol .....	155
Remote Dual Control with Brackets .....	173
Remote Bracket Dual Control for Modulite Seat .....	158
Remote IDC Dual Control with Brackets .....	159
ACS Cables & Accessories .....	174
Bus-Cables ACS .....	161
Light Cable ACS .....	162
T-Adapter - Modulite Seat .....	175
Programming Kits .....	164
Magnetkey for Remotes .....	165
Shark Electronic .....	176
Shark Electronic Modules .....	177



Yes, you can.®

## TABLE OF CONTENTS

---

Power Modul .....	178
Bracket for Power Modul .....	179
Light Board .....	132
Shark Remote & Brackets .....	180
Remote Shark with Brackets .....	181
Remote Bracket Standard .....	137
Remote Bracket Swing away & height adjust. ....	141
Remote Dual Control with Brackets .....	182
Remote Bracket Dual Control .....	183
Shark Cables & Accessories .....	184
Bus-Cables Shark .....	185
Cables Shark .....	186
Programming Kits .....	164

### Batteries, Cables & Chargers

Batteries, Cables & Chargers .....	187
Battery, Tray & Wiring - Wide Base .....	188
Battery, Tray & Wire Harness - Narrow Base .....	189
Charger .....	190

### Safety Options & Accessories

Lightsystem .....	191
Lightsystem LED .....	192
Lightsystem german Homologation .....	193
Rear Light Protector Kit .....	194
Reflective Sign .....	195
Adapter-Kit DAHL Docking Station - Wide Base .....	196
Adapter-Kit DAHL Docking Station - Narrow Base .....	197
Seat belts .....	198
Lap Belts (Modulite) .....	199
Lap Belt (Max Seat) .....	200
H-Belts (Modulite) .....	201
Rearview Mirror .....	202

### Seat Accessories

Head- / Neckrest .....	203
Headrest sidewise adjustable .....	204
Headrest with cheek support .....	205
Trunk Support - Modulite .....	206
Trunk Support, with extended Arm - Modulite .....	207

### Accessories

Cane holder .....	208
Cane holder (Modulite: backrest frame) .....	209
Cane holder (Modulite: back posts man recliner) .....	210
Cane holder (Modulite: back posts pow recliner) .....	211
Cane holder (MAX Seat) .....	212
Trays .....	213
Hour Counter .....	215
KLICKfix® bag & brackets .....	216





**Yes, you can.<sup>®</sup>**

## **TABLE OF CONTENTS**

---

KLICKfix <sup>®</sup> bag .....	217
KLICKfix <sup>®</sup> bracket for T-Armrest .....	218
KLICKfix <sup>®</sup> bracket for Fixed Armrest .....	219
KLICKfix <sup>®</sup> bracket for Inrigged/Flip-Up/Following Armrest (Modulite) .....	220
T-Nuts for Accessories Modulite Seat .....	221



## User Manuals TDX-SP2

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	CS SP1580120	User Manual TDX-SP2 - Czech		1
	DA SP1580070	User Manual TDX-SP2 - Danish		1
	DE SP1578996	User Manual TDX-SP2 - German		1
	EN SP1578997	User Manual TDX-SP2 - English		1
	ES SP1580066	User Manual TDX-SP2 - Spanish		1
	FI SP1580069	User Manual TDX-SP2 - Finnish		1
	FR SP1580064	User Manual TDX-SP2 - French		1
	IT SP1580065	User Manual TDX-SP2 - Italian		1
	LT SP1580121	User Manual TDX-SP2 - Lithuanian		1
	NL SP1580067	User Manual TDX-SP2 - Dutch		1
	NO SP1580072	User Manual TDX-SP2 - Norwegian		1
	PL SP1580122	User Manual TDX-SP2 - Polish		1
	PT SP1580068	User Manual TDX-SP2 - Portuguese		1
	RU SP1580123	User Manual TDX-SP2 - Russian		1
	SL SP1580124	User Manual TDX-SP2 - Slovenian		1
	SV SP1580071	User Manual TDX-SP2 - Swedish		1

## User Manuals Shark2

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1451346	User Manual Shark II German		1
	FR 1451347	User Manual Shark II French		1
	SV 1451348	User Manual Shark II Swedish		1
	DA 1451349	User Manual Shark II Danish		1
	NO 1451350	User Manual Shark II Norwegian		1
	NL 1451351	User Manual Shark II Dutch		1
	ES 1451352	User Manual Shark II Spanish		1
	PT 1451353	User Manual Shark II Portuguese		1
	IT 1451354	User Manual Shark II Italian		1
	FI 1451355	User Manual Shark II Finnish		1
	EN 1451356	User Manual Shark II English		1
	SL 1451584	User Manual Shark II Slovenian		1
	CS SP1533297	User Manual Shark2 CS		1
	ET SP1533298	User Manual Shark2 EE		1
	HU SP1533299	User Manual Shark2 HU		1
	LT SP1533300	User Manual Shark2 LT		1
	PL SP1533301	User Manual Shark2 PL		1
	BG SP1546061	User Manual Shark II Bulgarian		1
	RU SP1570186	User Manual Shark II Russian		1
	HR SP1574028	User Manual Shark II Croatian		1
	LV SP1576075	User Manual Shark II Latvian		1

**User Manuals REM A & REM B**

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DA 1529171	User Manual REM A & B Dansk		1
	DE 1529172	User Manual REM A & B German		1
	EN 1529173	User Manual REM A & B English		1
	ES 1529174	User Manual REM A & B Spanish		1
	FI 1529228	User Manual REM A & B Finsk		1
	FR 1529229	User Manual REM A & B French		1
	IT 1529230	User Manual REM A & B Italian		1
	NL 1529231	User Manual REM A & B Dutch		1
	NO 1529232	User Manual REM A & B Norsk		1
	PT 1529233	User Manual REM A & B Portuguese		1
	SV 1529234	User Manual REM A & B Swedish		1
	HR SP1533258	User Manual REM A & B Croatian		1
	CS SP1533259	User Manual REM A & B Czech		1
	ET SP1533265	User Manual REM A & B Estonian		1
	HU SP1533266	User Manual REM A & B Hungarian		1
	LT SP1533267	User Manual REM A & B Lithuanian		1
	PL SP1533268	User Manual REM A & B Polish		1
	SL SP1533269	User Manual REM A & B Slovenian		1
	RO SP1533270	User Manual REM A & B Romanian		1

## User Manuals REM24SD

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1436142	User Manual REM24SD German		1
	FR 1436143	User Manual REM24SD French		1
	EN 1436144	User Manual REM24SD English		1
	SV 1436145	User Manual REM24SD Swedish		1
	DA 1436146	User Manual REM24SD Danish		1
	NO 1436147	User Manual REM24SD Norwegian		1
	NL 1436148	User Manual REM24SD Dutch		1
	ES 1436149	User Manual REM24SD Spanish		1
	PT 1436150	User Manual REM24SD Portuguese		1
	IT 1436385	User Manual REM24SD Italian		1
	FI 1436386	User Manual REM24SD Finnish		1
	HR SP1533251	User Manual REM24SD Croatian		1
	CS SP1533252	User Manual REM24SD Czech		1
	ET SP1533253	User Manual REM24SD Estonian		1
	HU SP1533254	User Manual REM24SD Hungarian		1
	LT SP1533255	User Manual REM24SD Lithuanian		1
	PL SP1533256	User Manual REM24SD Polish		1
	SL SP1533257	User Manual REM24SD Slovenian		1
	LV SP1543152	User Manual REM24SD Latvian		1

## User Manuals REM550

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DA 1528088	User Manual REM550 Danish		1
	DE 1528089	User Manual REM550 German		1
	EN 1528090	User Manual REM550 English		1
	ES 1528091	User Manual REM550 Spanish		1
	FI 1528092	User Manual REM550 Finsk		1
	FR 1528093	User Manual REM550 French		1
	IT 1528094	User Manual REM550 Italiano		1
	NL 1528095	User Manual REM550 Dutch		1
	NO 1528096	User Manual REM550 Norsk		1
	PT 1528097	User Manual REM550 Portuguese		1
	SV 1528098	User Manual REM550 Swedish		1
	HR SP1533279	User Manual REM550 Croatian		1
	CS SP1533280	User Manual REM551 Czech		1
	ET SP1533281	User Manual REM552 Estonian		1
	HU SP1533282	User Manual REM553 Hungarian		1
	LT SP1533283	User Manual REM554 Lithuanian		1
	PL SP1533284	User Manual REM555 Polish		1
	SL SP1533285	User Manual REM556 Slovenian		1

## User Manuals G90

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	1523674	User Manual G90 English		1



## User Manuals G91

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1536441	User Manual G91 German		1
	DA 1537929	User Manual G91 Danish		1
	NL 1537930	User Manual G91 Dutch		1
	EN 1537931	User Manual G91 English		1
	FI 1537932	User Manual G91 Finnish		1
	FR 1537933	User Manual G91 French		1
	IT 1537934	User Manual G91 Italian		1
	NO 1537935	User Manual G91 Norwegian		1
	PT 1537936	User Manual G91 Portuguese		1
	ES 1537937	User Manual G91 Spanish		1
	SV 1537938	User Manual G91 Swedish		1

## User Manuals REM34 & REM41 Tray/Centre Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DA	SP1538022	User Manual REM41 Tray Control Danish		1
EN	SP1538023	User Manual REM41 Tray Control English		1
ES	SP1538024	User Manual REM41 Tray Control Spanish		1
FR	SP1538025	User Manual REM41 Tray Control French		1
IT	SP1538026	User Manual REM41 Tray Control Italian		1
NL	SP1538027	User Manual REM41 Tray Control Dutch		1
NO	SP1538028	User Manual REM41 Tray Control Norwegian		1
PT	SP1538029	User Manual REM41 Tray Control Portuguese		1
SV	SP1538030	User Manual REM41 Tray Control Swedish		1
CS	SP1538031	User Manual REM41 Tray Control Czech		1
SL	SP1538032	User Manual REM41 Tray Control Slovenian		1
DE	SP1541402	User Manual REM41 Tray Control German		1

## User Manuals IDC Dual Control

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
	DE 1521480	User Manual Dual-Control German		1
	EN 1521481	User Manual Dual-Control English		1
	FR 1521482	User Manual Dual-Control French		1
	NL 1521483	User Manual Dual-Control Dutch		1
	NO 1521484	User Manual Dual-Control Norwegian		1
	SV 1521485	User Manual Dual-Control Swedish		1
	DA 1521486	User Manual Dual-Control Danish		1
	IT 1522252	User Manual Dual-Control Italian		1
	PT 1522254	User Manual Dual-Control Portuguese		1
	ES 1522255	User Manual Dual-Control Spanish		1

## User Manuals ACS Handprogrammer

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DE	1419316	User Manual Handheld Programmer German		1
EN	1419317	User Manual Handheld Programmer English		1

# User Manual Proximity Head Remote DSL-105

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1555731	User Manual Proximity Head Remote DSL-105 English		1
DE	SP1555732	User Manual Proximity Head Remote DSL-105 German		1
FR	SP1555733	User Manual Proximity Head Remote DSL-105 French		1
DA	SP1555839	User Manual Proximity Head Remote DSL-105 Danish		1
ES	SP1555840	User Manual Proximity Head Remote DSL-105 Spanish		1
FI	SP1555841	User Manual Proximity Head Remote DSL-105 Finnish		1
IT	SP1555842	User Manual Proximity Head Remote DSL-105 Italian		1
NL	SP1555843	User Manual Proximity Head Remote DSL-105 Dutch		1
NO	SP1555844	User Manual Proximity Head Remote DSL-105 Norwegian		1
PT	SP1555845	User Manual Proximity Head Remote DSL-105 Portuguese		1
SV	SP1555846	User Manual Proximity Head Remote DSL-105 Swedish		1

## User Manuals Headcontrol

languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
DE	SP1493028	User Manual Headrest Headset German		1
EN	SP1493029	User Manual Headrest Headset English		1
NL	SP1493030	User Manual Headrest Headset Dutch		1

## User Manuals iPortal


languages.io

BG	Български	FR	Français	NL	Nederlands	SV	Svenska
CS	Čeština	HE	עברית	NO	Norsk	UK	Українська
DA	Dansk	HR	Hrvatski	PL	Polski		
DE	Deutsch	HU	Magyar	PT	Português		
EN	English	IT	Italiano	RU	Русский		
ES	Español	JA	日本語	RO	Română		
ET	Eesti	LV	Latviešu	SK	Slovenčina		
FI	Suomi	LT	Lietuvių	SL	Slovenščina		

Pos.	Part number	Description	Valid from → until	Qty
EN	SP1548231	User Manual iPortal english		1
DA	SP1552493	User Manual iPortal danisch		1
FR	SP1552494	User Manual iPortal french		1
FI	SP1552512	User Manual iPortal finnish		1
NO	SP1552513	User Manual iPortal norwegian		1
SV	SP1552533	User Manual iPortal swedish		1
IT	SP1552534	User Manual iPortal Italian		1
DE	SP1552535	User Manual iPortal german		1
SL	SP1552703	User Manual iPortal slovenian		1
NL	SP1559807	User Manual iPortal Dutch		1
PT	SP1567967	User Manual iPortal Portuguese		1
ES	SP1568026	User Manual iPortal Spanish		1

# Chassis

SPCTDX\_SP210.iso

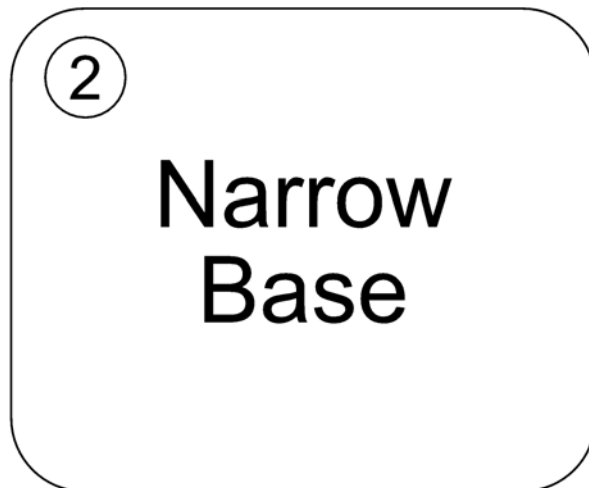

Pos.	Part number	Description	Valid from → until	Qty
1	-	Chassis Wide Base		1
2	-	Chassis Narrow Base		1




**Chassis Wide Base**

Pos.	Part number	Description	Valid from → until	Qty
-		Battery Box Hardware		1
-		Walking Beam		1
-		Rear Pivot Wide Base		1
-		Shrouds Wide Base		1
-		Low Rider Option		1

# Battery Box Hardware


1583232.00

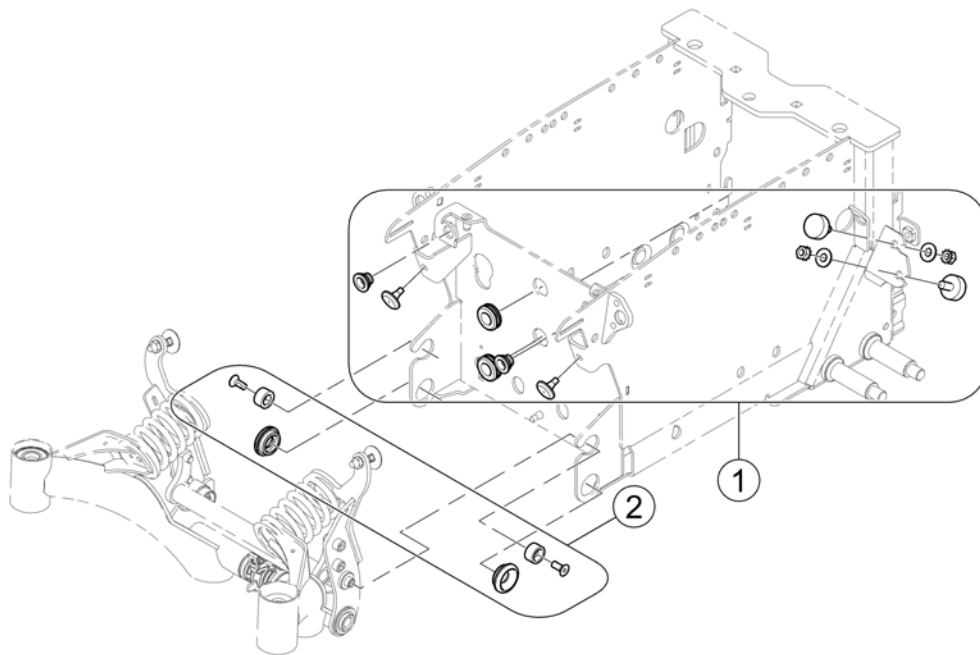

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523179	Kit, Battery Box, Rubber Bumper & Grommet Hardware		1
2	SP1523193	Kit, Rear Pivot Assembly Mounting Hardware		1

# Walking Beam


1583259.000

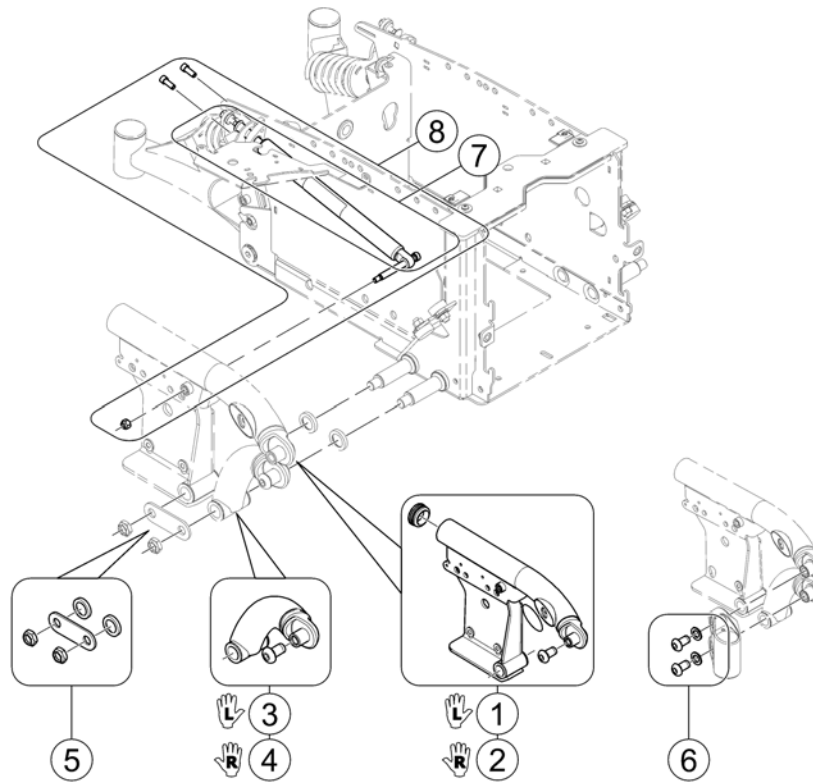

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179092	Kit, Walking Beam Assy Black, SSD LH		1
2	SP1179091	Kit, Walking Beam Assy Black, SSD RH		1
3	SP1178814	Link Assy, Lower Pivot Black LH		1
4	SP1178813	No description available.		1
5	SP1523168	Kit, Lower Pivot Link / 4-Pole Motor Mounting Hardware		1
6	SP1523176	Kit, Front Head Tube Mounting Hardware		1
7	SP1523158	Cylinder, Locking Gas (400 Lb. User Weight Limit)		1
8	SP1179253	Kit, Locking Gas Cylinder & Mounting Hardware		1

## Rear Pivot Wide Base


1543285.000

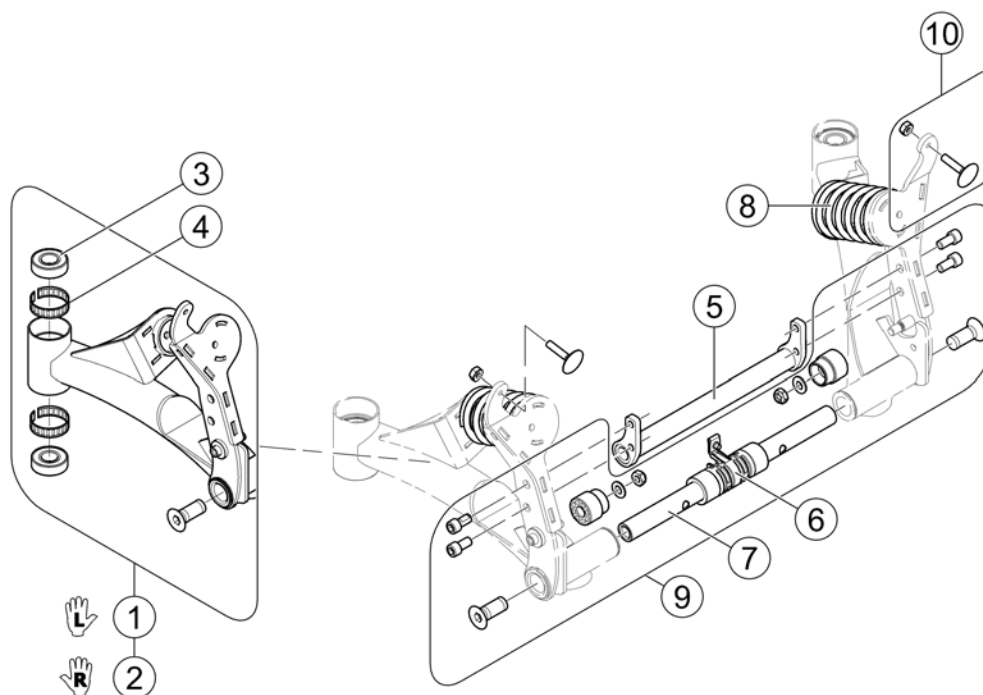

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179094	Kit, Rear Swing Arm Assy w/ Hardware, black, LH		1
2	SP1179093	Kit, Rear Swing Arm Assy w/ Hardware, black, RH		1
3	SP40117X006	Bearing (5/8 ")		1
4	SP1110717	Ring, Tolerance		1
5	SP1176365	Crossmember, Pivot, black, Widebase		1
6	SP1523166	Bushing Assembly, Center		1
7	SP1155020	Bar, Pivot - Wide Base		1
8	SP1579643	Spring - silver		1
9	SP1523177	Kit, Pivot Assembly Hardware		1
10	SP1151949	Kit, Elevator Bolt w/ Keps Nut (1/4-20)		1

**Shrouds Wide Base**


1583238.00

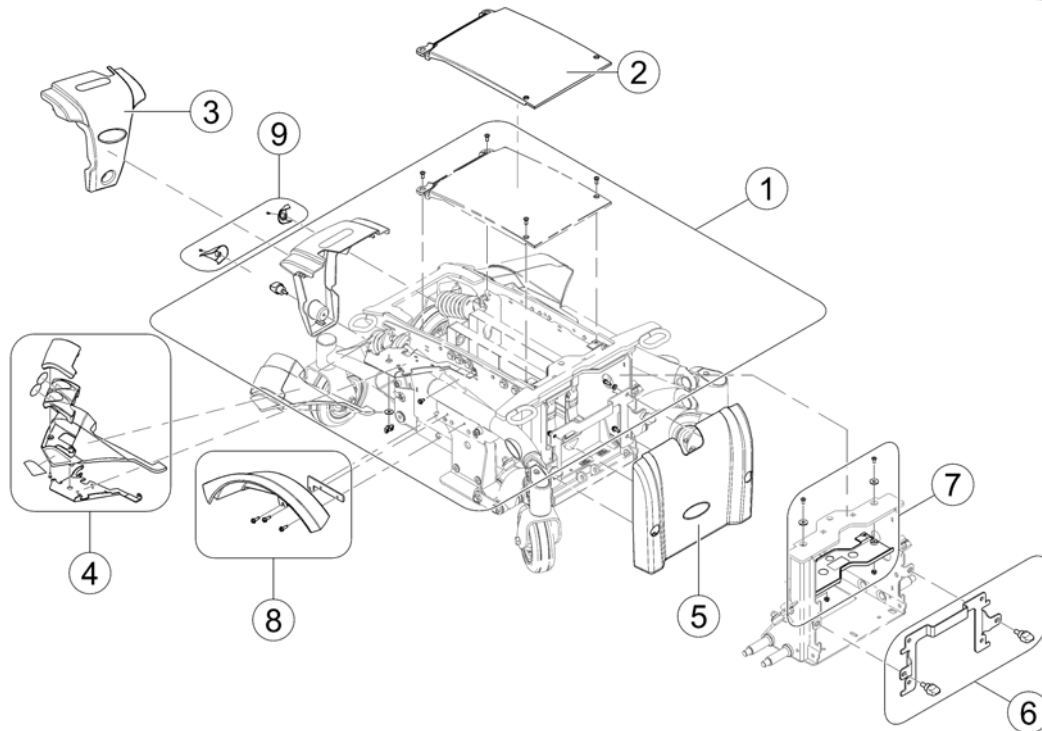

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179326	Kit, Shroud Hardware		1
2	SP1174636	Shroud, Center, black - Wide Base		1
3	-	Rear Shrouds - Wide Base		1
4	-	Bezel Shroud		1
5	-	Front Shroud - Wide Base		1
6	SP1179898	Front Shroud, Bracket & Hardware - Wide Base		1
7	SP1179739	Isolation Shroud Assy - Wide Base		1
8	-	Fender		1
9	-	Rear Caps		1

## Rear Shrouds - Wide Base


1583254 rev

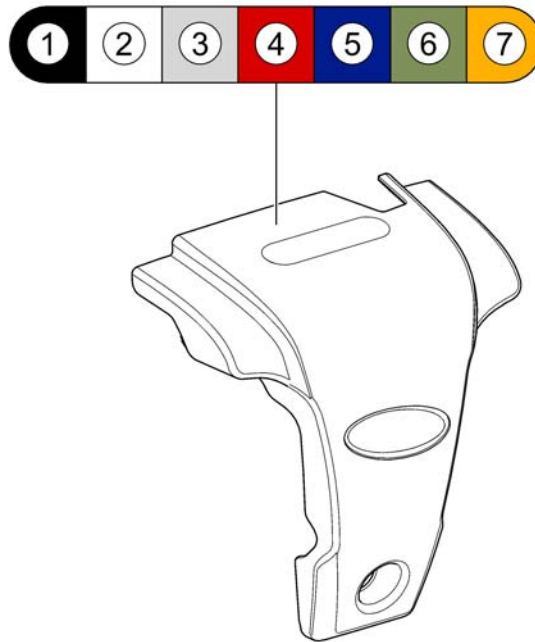

Pos.	Part number	Description	Valid from → until	Qty
1	SP1176568	Rear Shroud, Wet Black - Wide Base		1
2	SP1176569	Rear Shroud, Arctic White - Wide Base		1
3	SP1176570	Rear Shroud, Electric Silver - Wide Base		1
4	SP1176571	Rear Shroud, Electric Red - Wide Base		1
5	SP1176572	Rear Shroud, Electric Blue - Wide Base		1
6	SP1176573	Rear Shroud, Aviator Green - Wide Base		1
7	SP1176574	Rear Shroud, Desert Orange - Wide Base		1

# Bezel Shroud


1583256 Rev

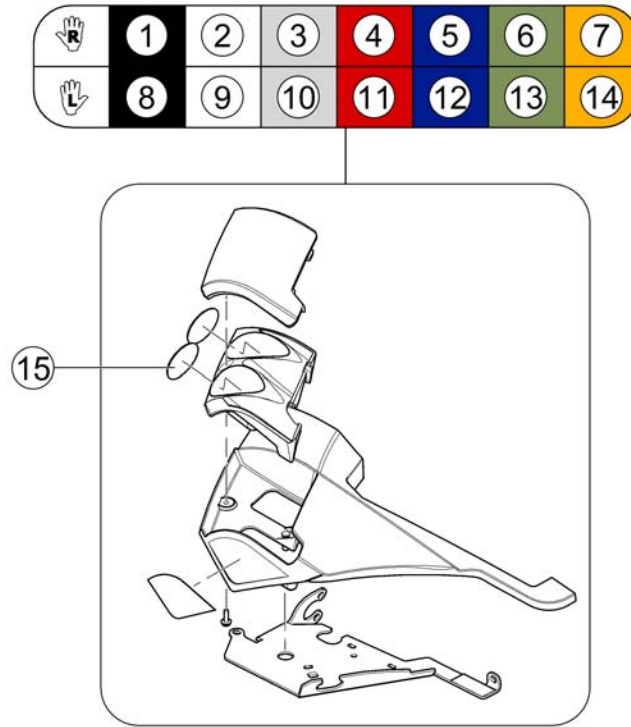

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179237	Coloured Bezel Shroud RH w/ Mount Wet Black		1
2	SP1179238	Coloured Bezel Shroud RH w/ Mount Artic White		1
3	SP1179239	Coloured Bezel Shroud RH w/ Mount Electric Silver		1
4	SP1179240	Coloured Bezel Shroud RH w/ Mount Electric Red		1
5	SP1179241	Coloured Bezel Shroud RH w/ Mount Electric Blue		1
6	SP1179242	Coloured Bezel Shroud RH w/ Mount Aviator Green		1
7	SP1179243	Coloured Bezel Shroud RH w/ Mount Desert Orange		1
8	SP1179244	Coloured Bezel Shroud LH w/ Mount Wet Black		1
9	SP1179245	Coloured Bezel Shroud LH w/ Mount Artic White		1
10	SP1179246	Coloured Bezel Shroud LH w/ Mount Electric Silver		1
11	SP1179247	Coloured Bezel Shroud LH w/ Mount Electric Red		1
12	SP1179248	Coloured Bezel Shroud LH w/ Mount Electric Blue		1
13	SP1179249	Coloured Bezel Shroud LH w/ Mount Aviator Green		1
14	SP1179250	Coloured Bezel Shroud LH w/ Mount Desert Orange		1
15	SP1540286	Reflector-Kit (4pcs), red, Dia. 28mm		1

## Front Shroud - Wide Base

1583258.iss

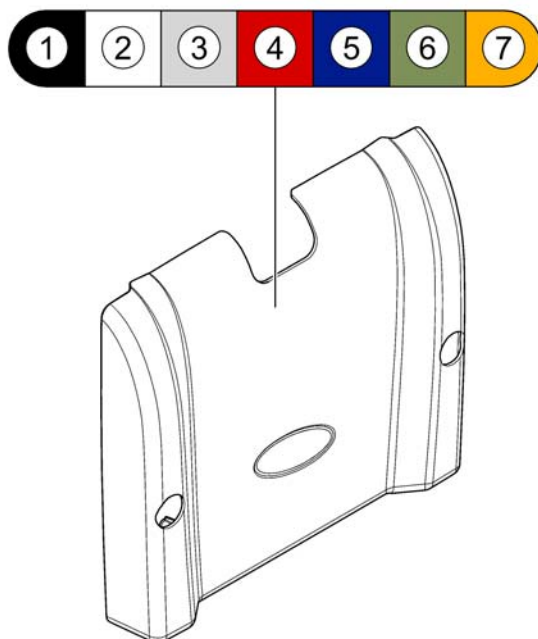


Pos.	Part number	Description	Valid from → until	Qty
1	SP1179717	Coloured Front Shroud WB Assy Wet Black		1
2	SP1189505	Coloured Front Shroud WB Assy Artic White		1
3	SP1189506	Coloured Front Shroud WB Assy Electric Silver		1
4	SP1179718	Coloured Front Shroud WB Assy Electric Red		1
5	SP1189507	Coloured Front Shroud WB Assy Electric Blue		1
6	SP1179719	Coloured Front Shroud WB Assy Aviator Green		1
7	SP1189508	Coloured Front Shroud WB Assy Desert Orange		1



# Fender


1583236 Rev

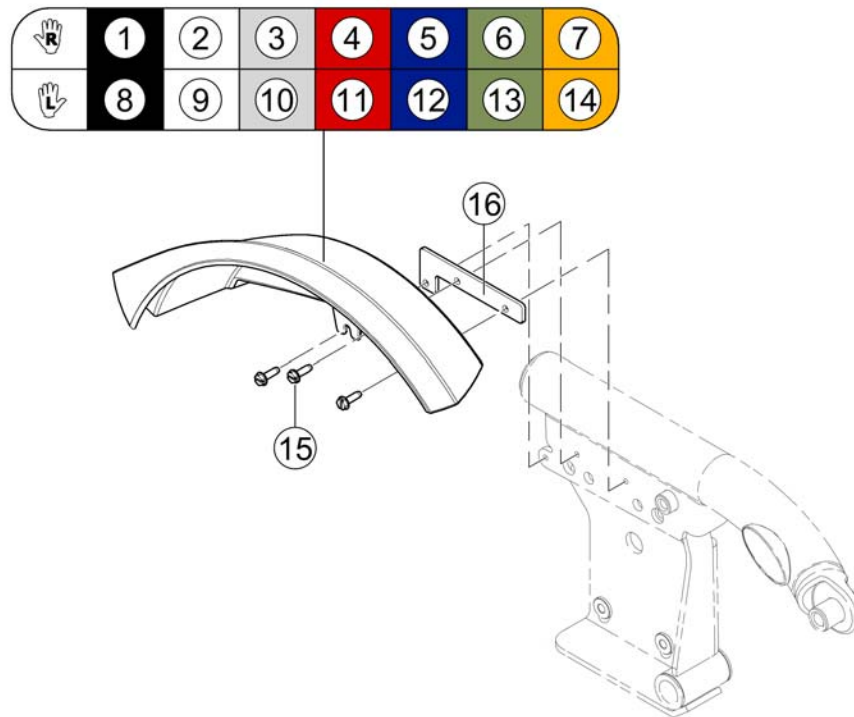

Pos.	Part number	Description	Valid from → until	Qty
1	SP1176696	Coloured Fender Shroud RH Assy Wet Black		1
2	SP1176697	Coloured Fender Shroud RH Assy Artic White		1
3	SP1176698	Coloured Fender Shroud RH Assy Electric Silver		1
4	SP1176699	Coloured Fender Shroud RH Assy Electric Red		1
5	SP1176700	Coloured Fender Shroud RH Assy Electric Blue		1
6	SP1176701	Coloured Fender Shroud RH Assy Aviator Green		1
7	SP1176702	Coloured Fender Shroud RH Assy Desert Orange		1
8	SP1176703	Coloured Fender Shroud LH Assy Wet Black		1
9	SP1176704	Coloured Fender Shroud LH Assy Artic White		1
10	SP1176705	Coloured Fender Shroud LH Assy Electric Silver		1
11	SP1176706	Coloured Fender Shroud LH Assy Electric Red		1
12	SP1176707	Coloured Fender Shroud LH Assy Electric Blue		1
13	SP1176708	Coloured Fender Shroud LH Assy Aviator Green		1
14	SP1176709	Coloured Fender Shroud LH Assy Desert Orange		1
15	SP1025451	Act.Tech Screw HexHead10-32		1
16	SP1179913	Spacer, Shroud, Black		1

## Rear Caps


1583287 Rev

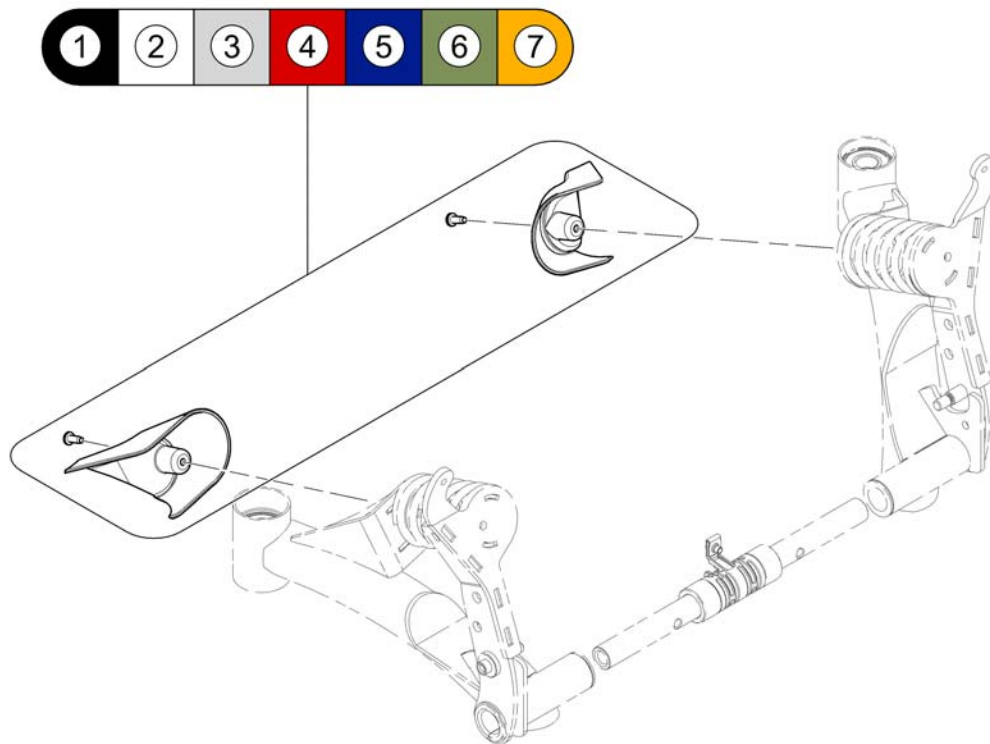

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179084	Kit, Coloured Rear Caps w/ Mount Wet Black		1
2	SP1179085	Kit, Coloured Rear Caps w/ Mount Artic White		1
3	SP1179086	Kit, Coloured Rear Caps w/ Mount Electric Silver		1
4	SP1179087	Kit, Coloured Rear Caps w/ Mount Electric Red		1
5	SP1179088	Kit, Coloured Rear Caps w/ Mount Electric Blue		1
6	SP1179089	Kit, Coloured Rear Caps w/ Mount Aviator Green		1
7	SP1179090	Kit, Coloured Rear Caps w/ Mount Desert Orange		1

# Low Rider Option

1587261 Rev

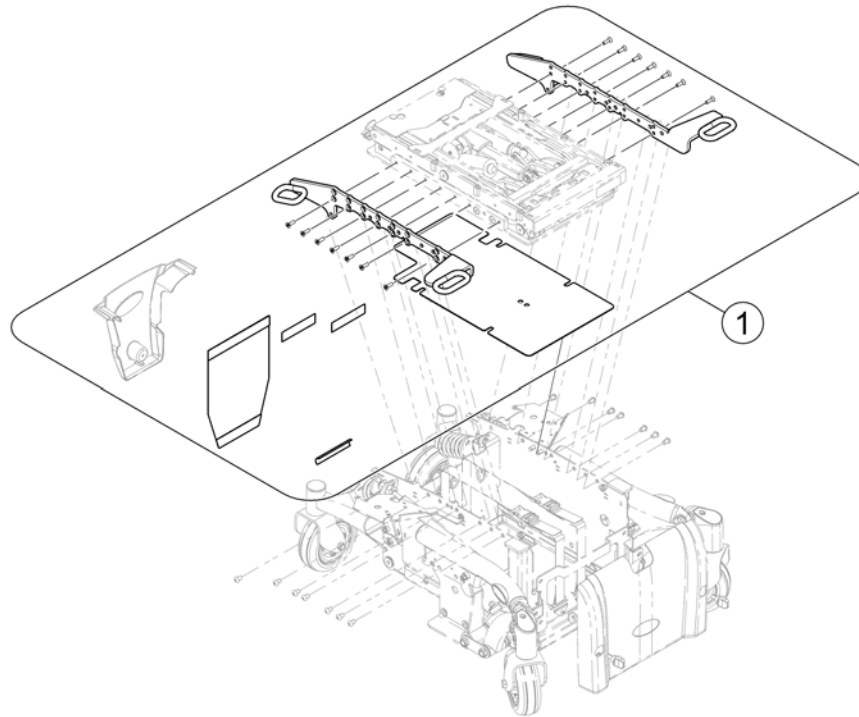


Pos.	Part number	Description	Valid from → until	Qty
-		The Low Rider Kit may not be used for retrofitting! !!!		0
1	SP1587260	Low Rider Kit		1

## Chassis Narrow Base

Pos.	Part number	Description	Valid from → until	Qty
-		Battery Box Hardware		1
-		Walking Beam		1
-		Rear Pivot Narrow Base		1
-		Shrouds Narrow Base		1

**Rear Pivot Narrow Base**


1543285 Rev

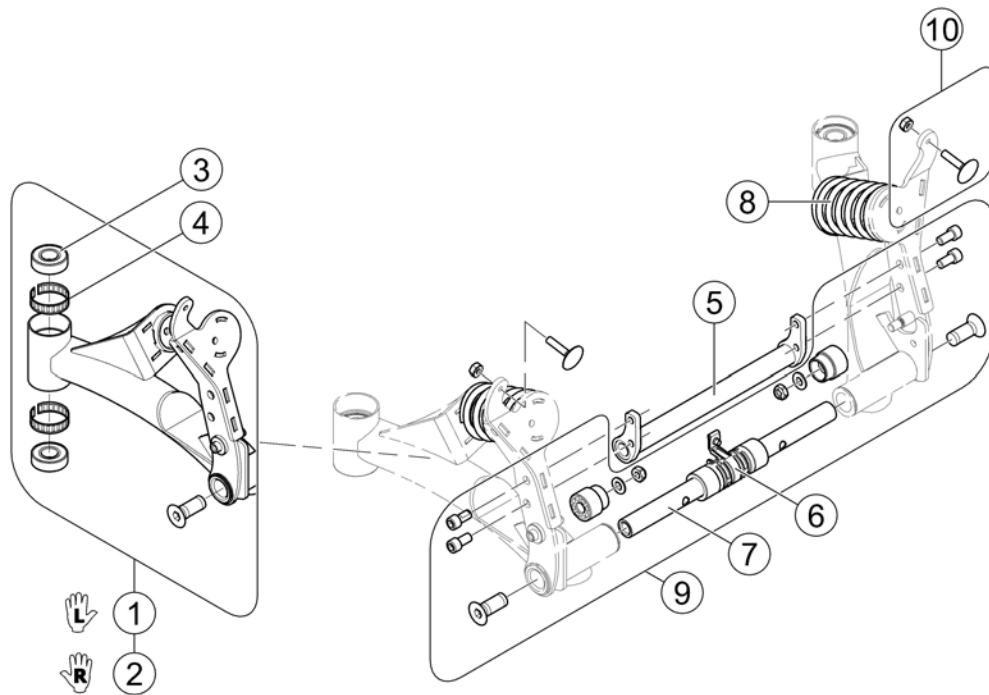

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179094	Kit, Rear Swing Arm Assy w/ Hardware, black, LH		1
2	SP1179093	Kit, Rear Swing Arm Assy w/ Hardware, black, RH		1
3	SP40117X006	Bearing (5/8 ")		1
4	SP1110717	Ring, Tolerance		1
5	SP1176604	Crossmember, Pivot, black - Narrow Base		1
6	SP1523166	Bushing Assembly, Center		1
7	SP1155018	No description available.		1
8	SP1579643	Spring - silver		1
9	SP1523177	Kit, Pivot Assembly Hardware		1
10	SP1151949	Kit, Elevator Bolt w/ Keps Nut (1/4-20)		1

# Shrouds Narrow Base

1583238.00

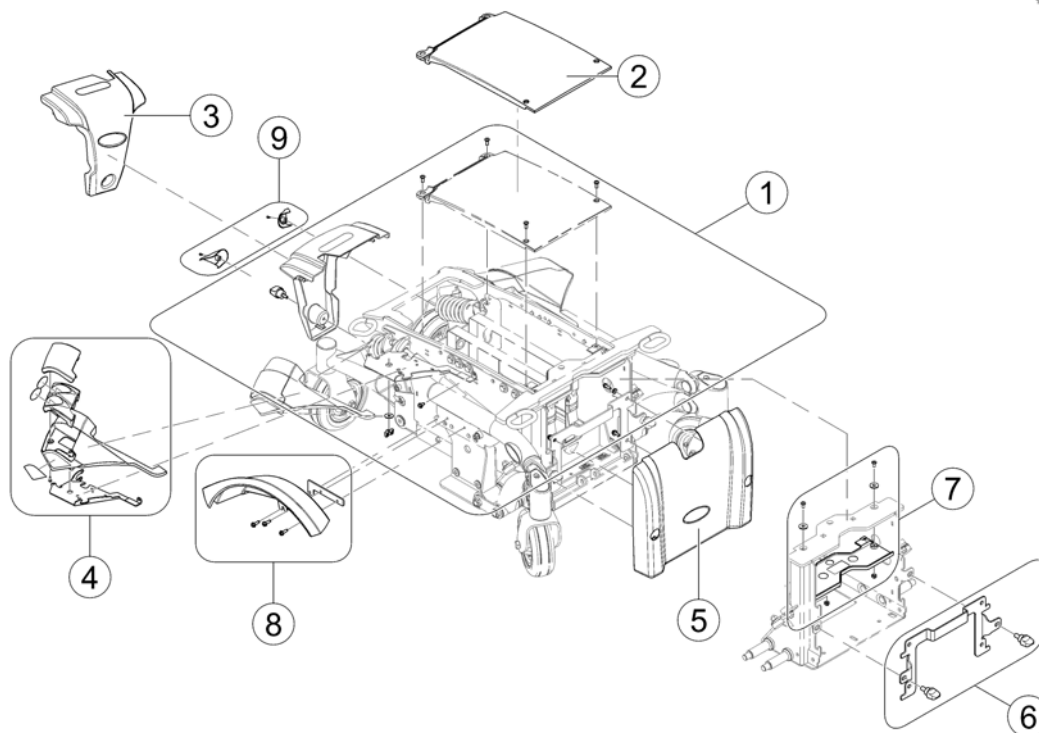

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179326	Kit, Shroud Hardware		1
2	SP1174634	Shroud, Center, black - Narrow Base		1
3	-	Rear Shrouds Narrow Base		1
4	-	Bezel Shroud		1
5	-	Front Shroud Narrow Base		1
6	SP1179897	Front Shroud, Bracket & Hardware - Narrow Base		1
7	SP1179738	Isolation Shroud Assy - Narrow Base		1
8	-	Fender		1
9	-	Rear Caps		1

## Rear Shrouds Narrow Base


1583254 rev



Pos.	Part number	Description	Valid from → until	Qty
1	SP1176547	Coloured Rear Shroud NB w/ Decals Wet Black		1
2	SP1176548	Coloured Rear Shroud NB w/ Decals Artic White		1
3	SP1176549	Coloured Rear Shroud NB w/Decals Electric Silver		1
4	SP1176550	Coloured Rear Shroud NB w/ Decals Electric Red		1
5	SP1176551	Coloured Rear Shroud NB w/ Decals Electric Blue		1
6	SP1176552	Coloured Rear Shroud NB w/ Decals Aviator Green		1
7	SP1176553	Coloured Rear Shroud NB w/ Decals Desert Orange		1


## Front Shroud Narrow Base

1583258.iss

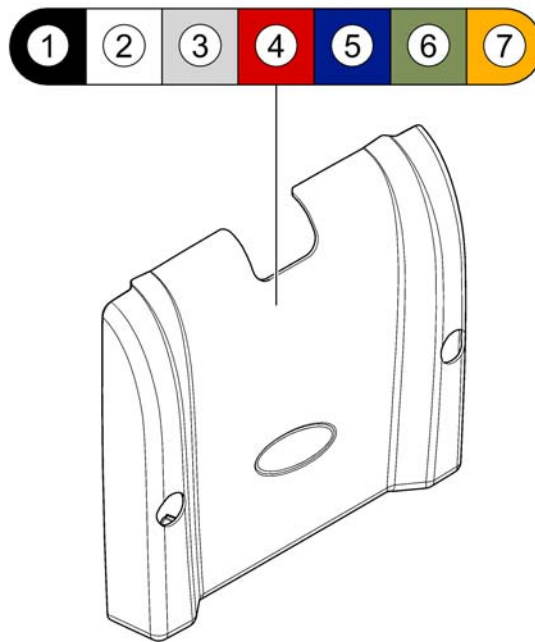

Pos.	Part number	Description	Valid from → until	Qty
1	SP1179714	Coloured Front Shroud NB Assy Wet Black		1
2	SP1189214	Coloured Front Shroud NB Assy Artic White		1
3	SP1189215	Coloured Front Shroud NB Assy Electric Silver		1
4	SP1179715	Coloured Front Shroud NB Assy Electric Red		1
5	SP1189216	Coloured Front Shroud NB Assy Electric Blue		1
6	SP1179716	Coloured Front Shroud NB Assy Aviator Green		1
7	SP1189217	Coloured Front Shroud NB Assy Desert Orange		1






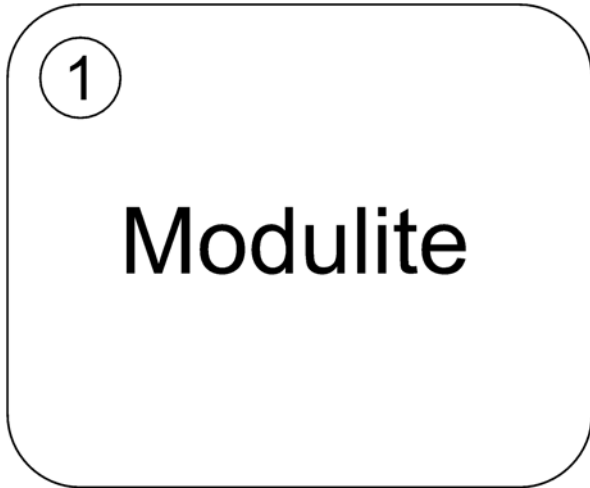

Pos.	Part number	Description	Valid from → until	Qty
1	-	Modulite Seat		1
2	-	Max Seat		1


# Modulite Seat

1582372.000

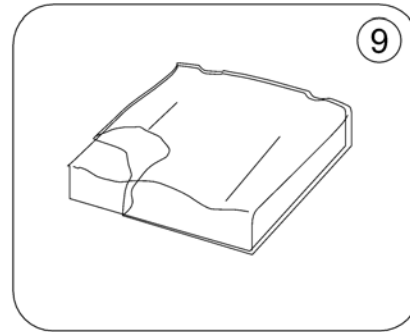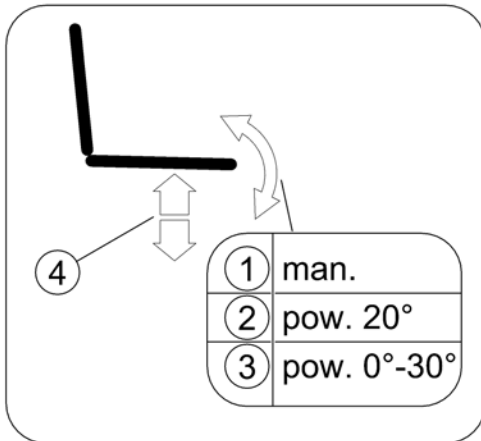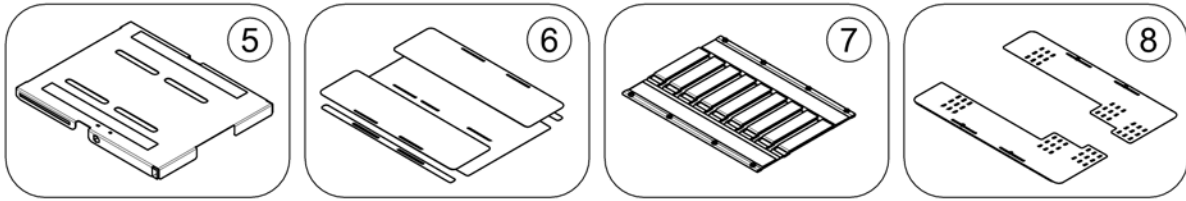

Pos.	Part number	Description	Valid from → until	Qty
1	-	man. settable Tilt [1091]		1
2	-	powered simple Tilt 20° [1092]		1
3	-	powered Tilt 0°-30° [1093]		1
4	-	Lifter [1094]		1
5	-	One Piece Seat Plate [2110]		1
6	-	Telescopic Seat Frame and Plate [2111]		1
7	-	Telescopic Seat Frame and Sling Seat [2112]		1
8	-	Telescopic Seat Frame and Universal Adapter [2109]		1
9	-	Seat Cushion		1


1577301 iso

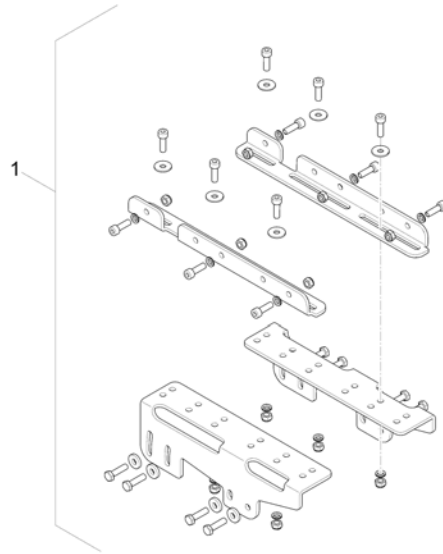

Pos.	Part number	Description	Valid from → until	Qty
1	SP1563229	Mounting Rails		1


# powered simple Tilt 20°

1582373.mo

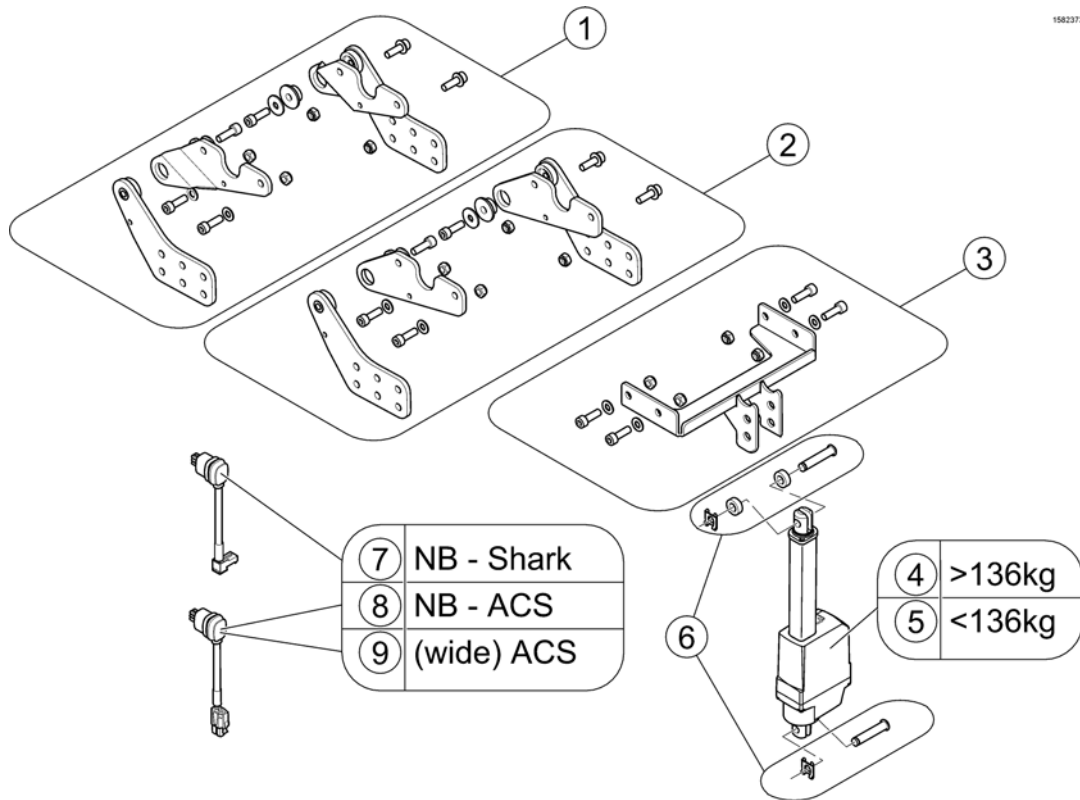

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582738	Pivot Bearing Back - Wide Base		1
2	SP1582736	Pivot Bearing Back - Narrow Base		1
3	SP1582731	Support Plate Front for Tilt		1
4	SP1556487	Actuator LA23 - stroke 135mm / 2500N		1
5	SP1581020	Actuator LA23 - stroke 135mm / 1800N		1
6	SP1582737	Mounting Hardware for Tilt		1
7	SP1552297	Actuator Cable LA23 Shark (~500mm)		1
8	SP1556449	Actuator Cable LA23 ACS2 (~400mm)		1
9	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1


157302.iss

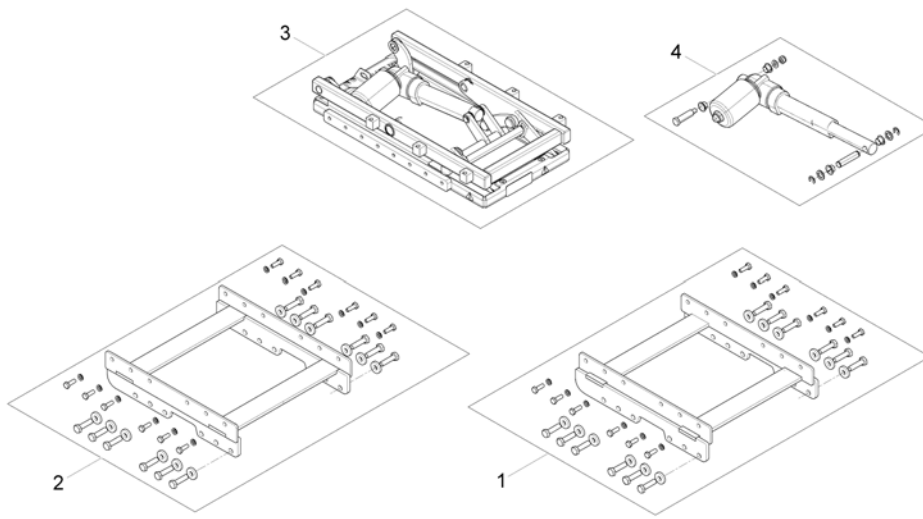

Pos.	Part number	Description	Valid from → until	Qty
1	SP1568426	Lifter- /Tilt Bracket for wide Base		1
2	SP1568424	Lifter- /Tilt Bracket for narrow Base		1
3	SP1558714	Tilt unit		1
4	SP1571809	Actuator		1
-	SP1570291	Adapter Cable		1

# Lifter


1577303 iso

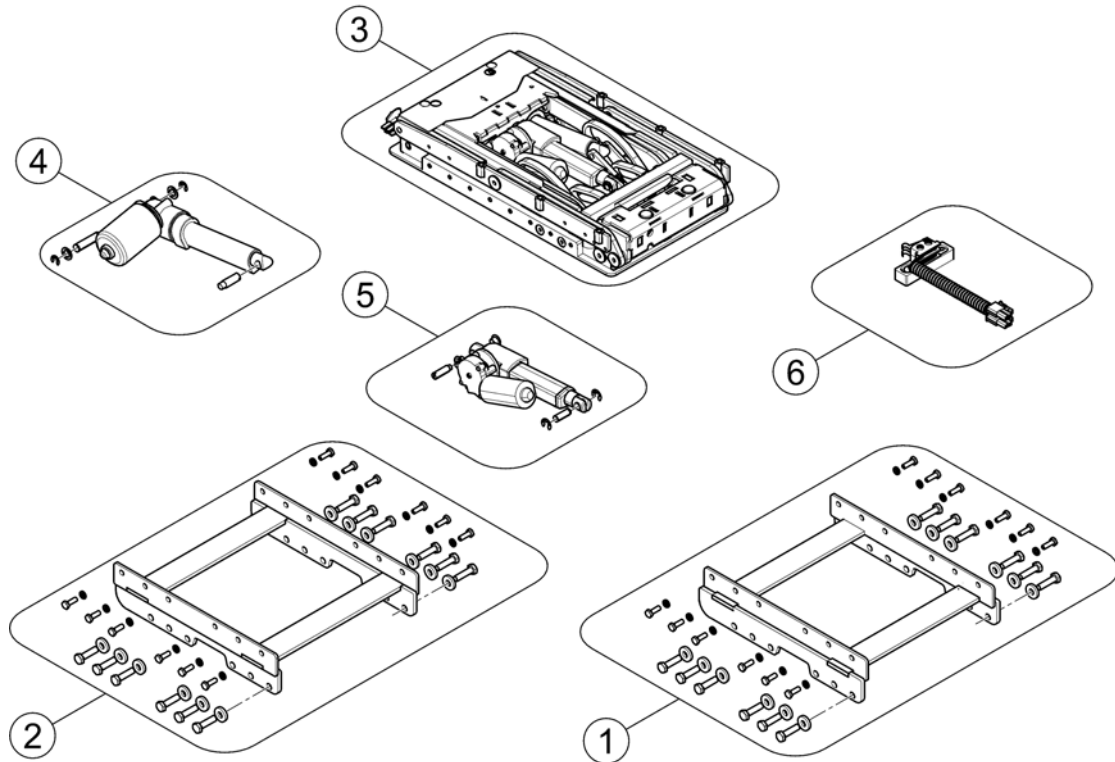

Pos.	Part number	Description	Valid from → until	Qty
-	SP1570291	Adapter Cable <i>(2x for wide Base / 1x for Narrow Base)</i>		1
-	SP1576062	Extension Cable <i>(only for narrow Base)</i>		1
-	SP1570294	Extension Cable		1
-	SP1533561	Light Cable w. DCI		1
1	SP1568426	Lifter- /Tilt Bracket for wide Base		1
2	SP1568424	Lifter- /Tilt Bracket for narrow Base		1
3	SP1558715	Lifter- / Tilt unit		1
4	SP1571823	Lifter actuator		1
5	SP1571822	Tilt actuator		1
6	SP1577762	Slow-Stop Switchcable with Mounting		1

# One Piece Seat Plate


1571064.000

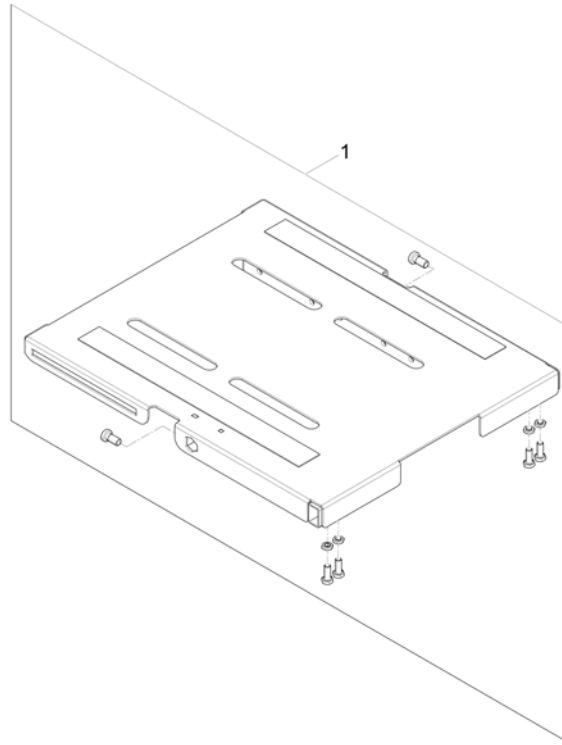

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571297	One Piece Seat Plate		1

# Telescopic Seat Frame and Plate

1572312 iso

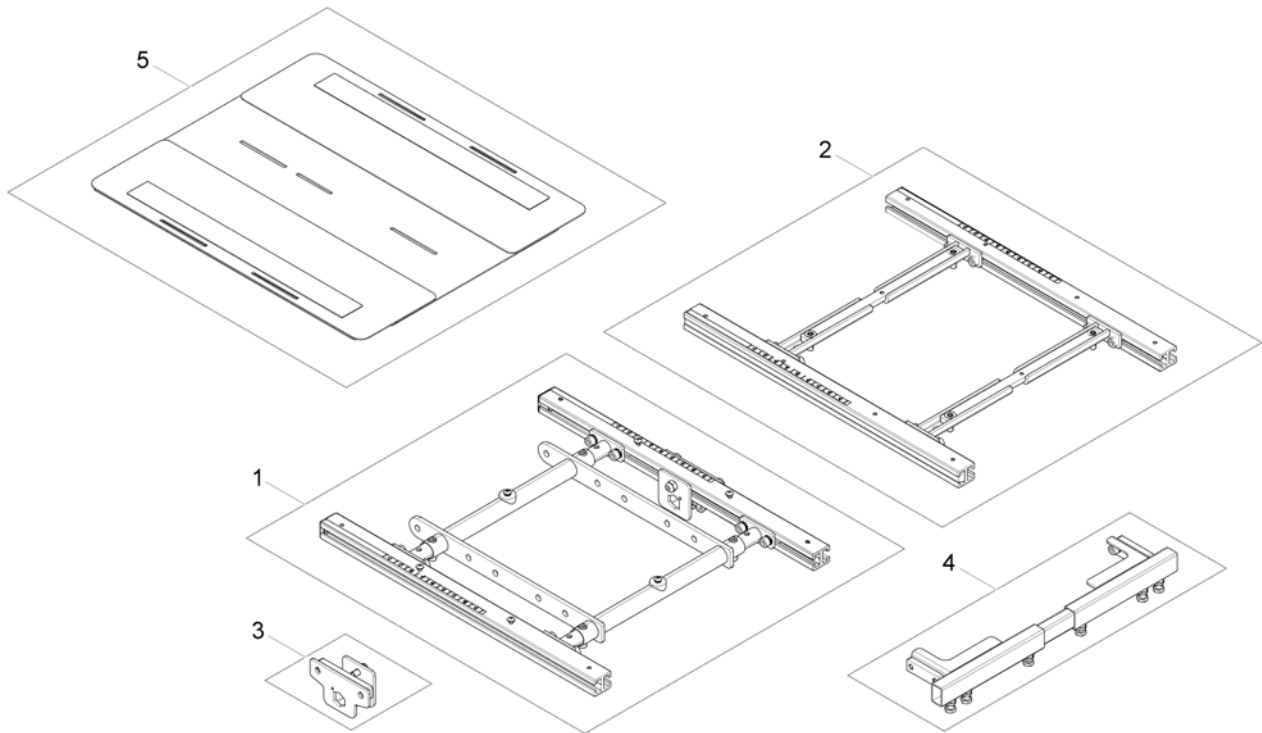


Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for man. /pow. 20° Tilt		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest Bracket		1
4	-	Legrest Support		1
5	-	Seat Plate		1



# Telescopic Seat Frame for man. /pow. 20° Tilt


1502402-001

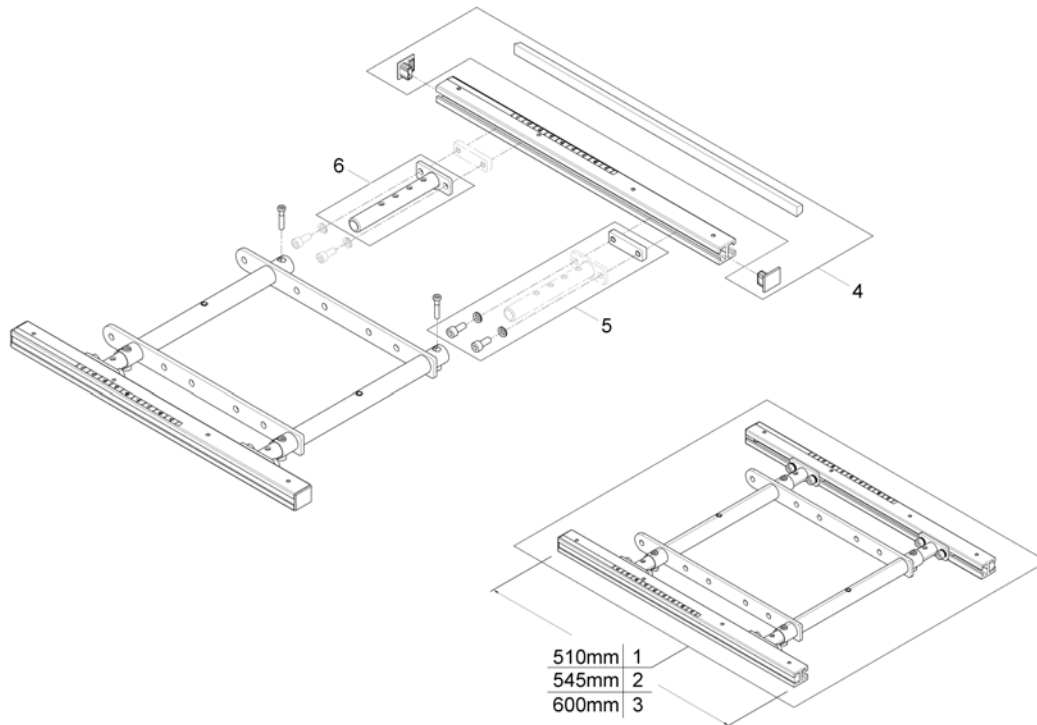

Pos.	Part number	Description	Valid from → until	Qty
-	SP1588443	Seat Rails 510mm (2pcs kit)		1
-	SP1588444	Seat Rails 545mm (2pcs kit)		1
-	SP1588445	Seat Rails 600mm (2pcs kit)		1
1	SP1572977	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582216	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1582739	Seatframe 600mm, cpl., pre set up to SW 43cm		1
4	SP1571810	Sealing strap with plugs		1
5	SP1571811	Groove stone with mounting parts		1
6	SP1554646	Adjustment Tube		1

# Telescopic Seat Frame for Tilt 30° / Lifter


1572403 iso

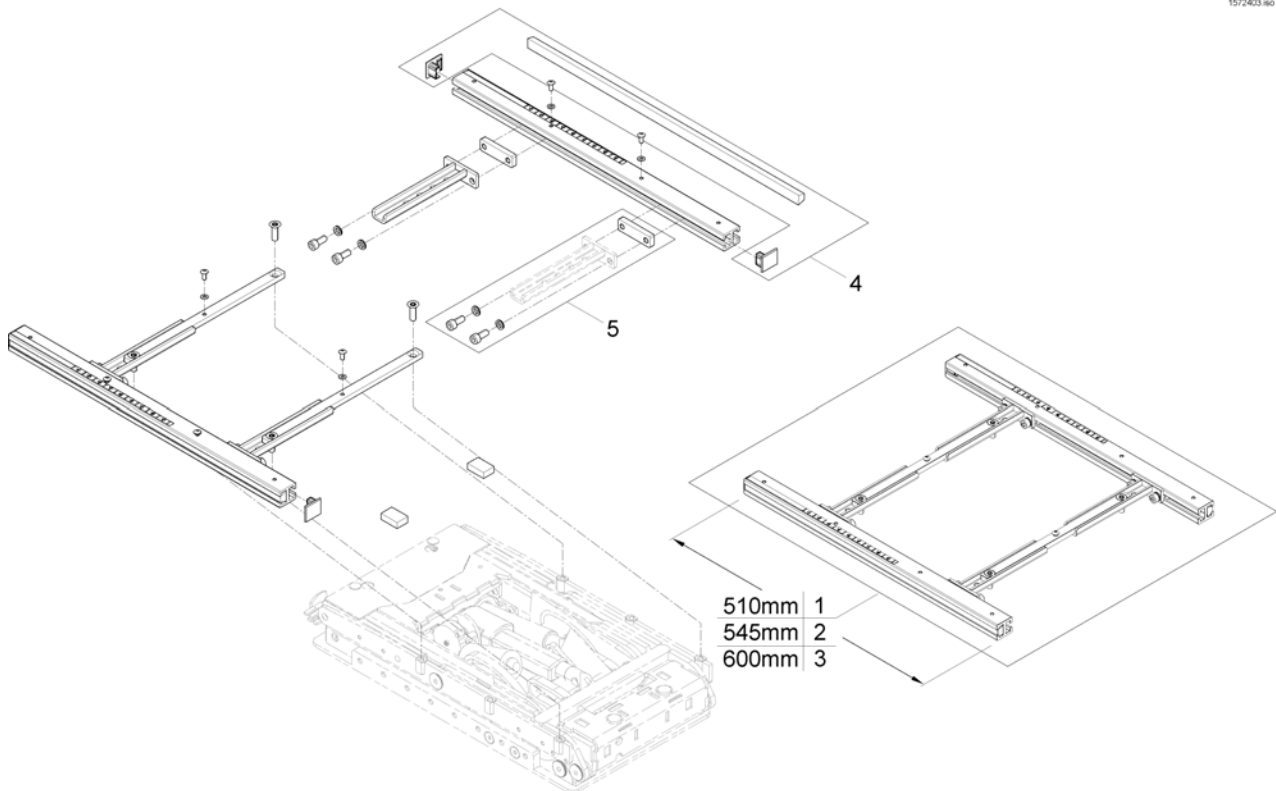

Pos.	Part number	Description	Valid from → until	Qty
-	SP1588443	Seat Rails 510mm (2pcs kit)		1
-	SP1588444	Seat Rails 545mm (2pcs kit)		1
-	SP1588445	Seat Rails 600mm (2pcs kit)		1
1	SP1572992	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582219	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1582740	Seatframe 600mm, pre set up to SW 430mm, cpl.		1
4	SP1571810	Sealing strap with plugs		1
5	SP1571811	Groove stone with mounting parts		1

# Armrest Bracket


1587804 Rev

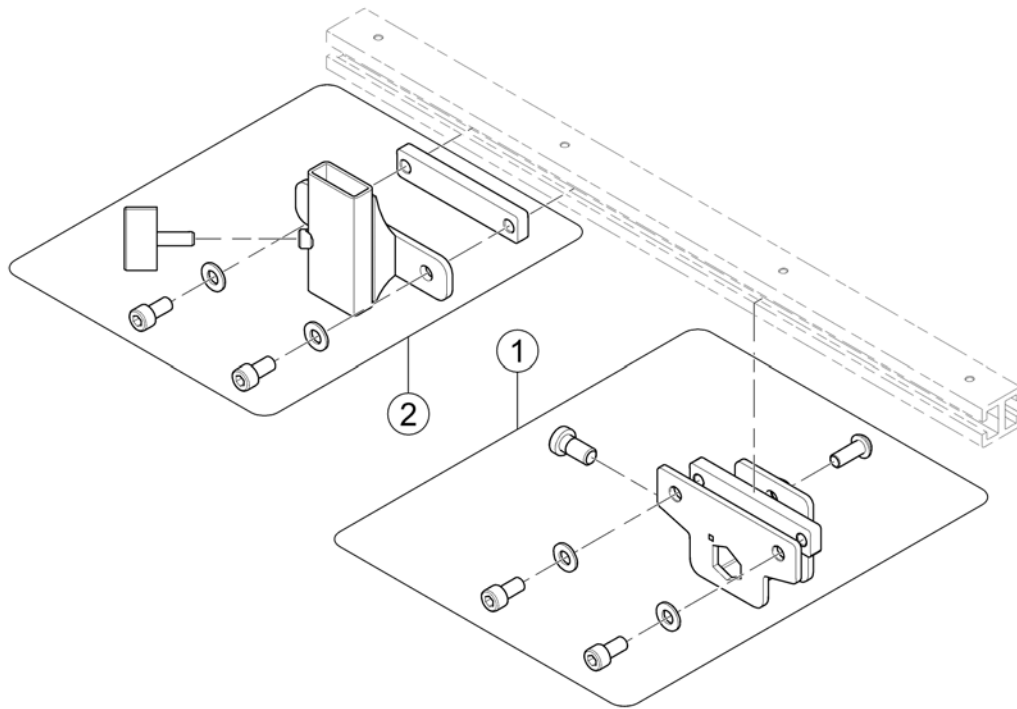

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570189	Armrest bracket		1
2	SP1585469	Armrest Bracket (especially for Low Rider)		1

# Legrest Support


1572021.000

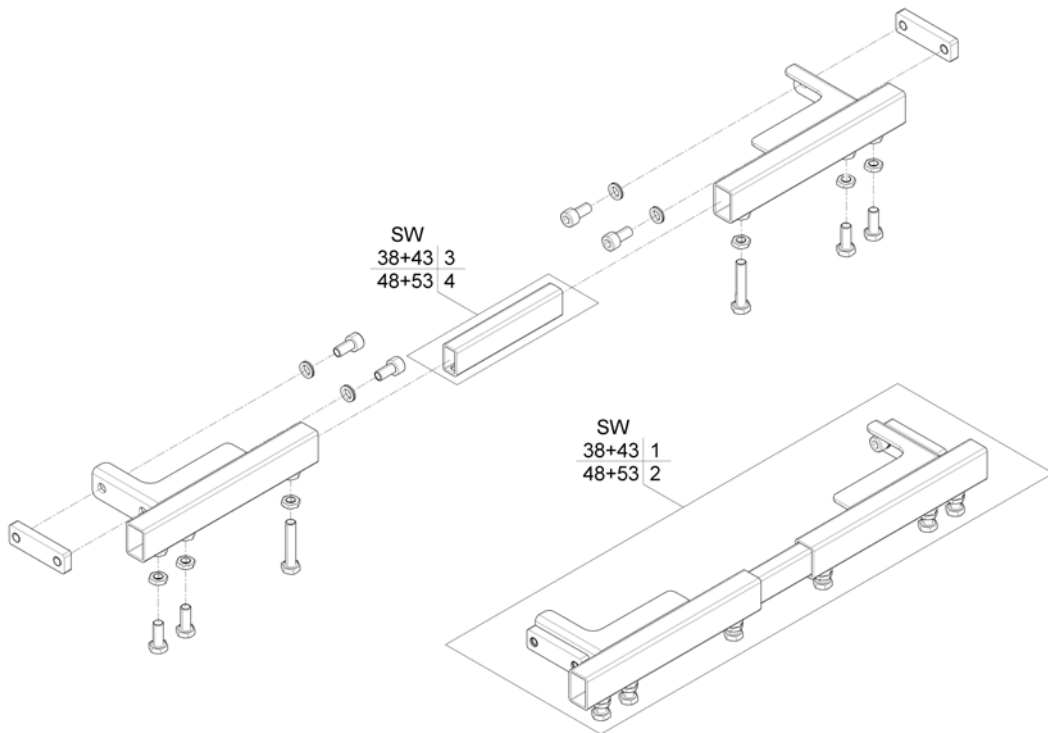

Pos.	Part number	Description	Valid from → until	Qty
1	SP1558935	Legrest Support SW38+43, cpl.		1
2	SP1559294	Legrest Support SW48+53, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

# Seat Plate


1582404.iso

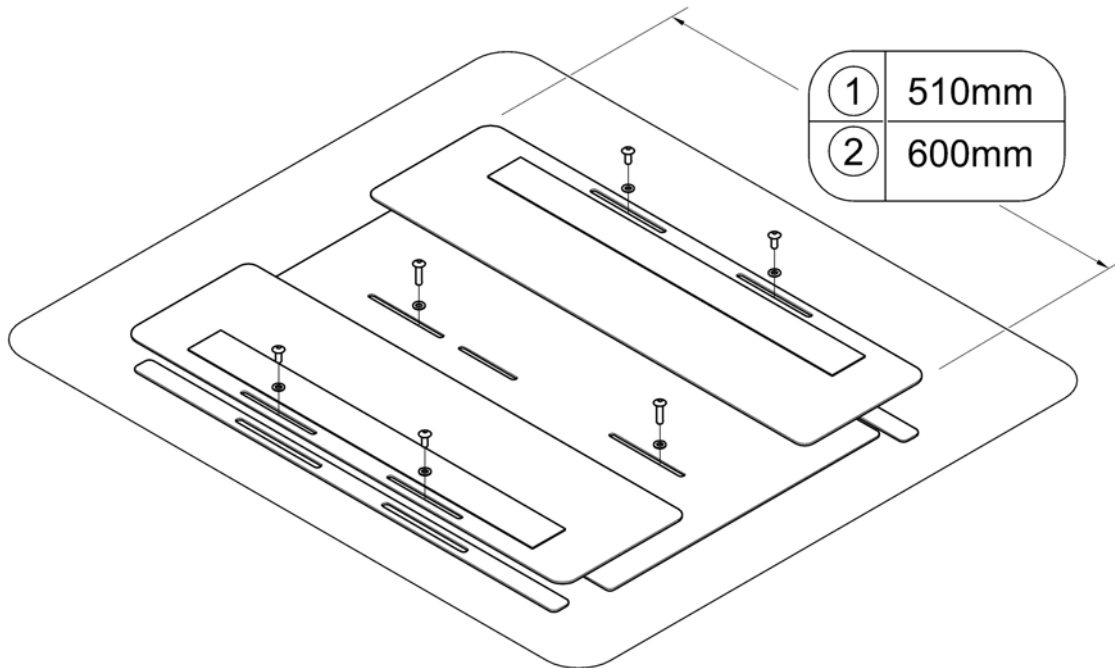

Pos.	Part number	Description	Valid from → until	Qty
1	SP1572199	Seat Plate 510mm, cpl.		1
2	SP1580756	Seat Plate 600mm, cpl.		1

# Telescopic Seat Frame and Sling Seat


1572311.00

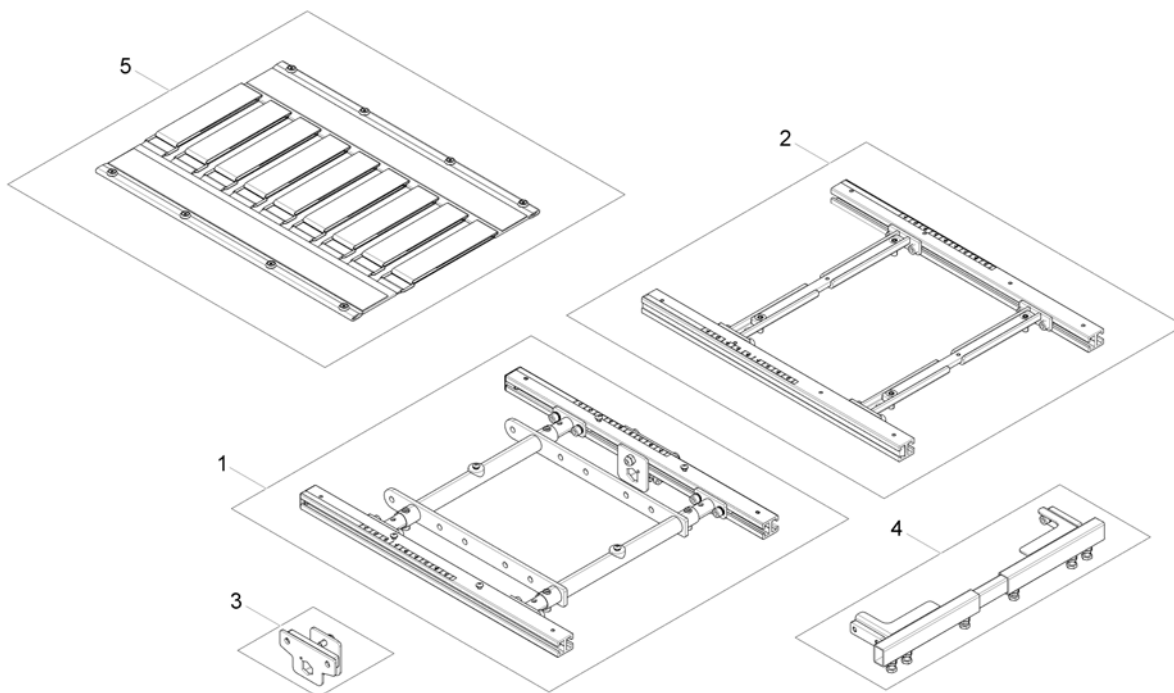

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for man. / pow. 20° Tilt		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest Bracket		1
4	-	Legrest Support		1
5	-	Sling Seat		1

# Telescopic Seat Frame for man. / pow. 20° Tilt


1582402\_00

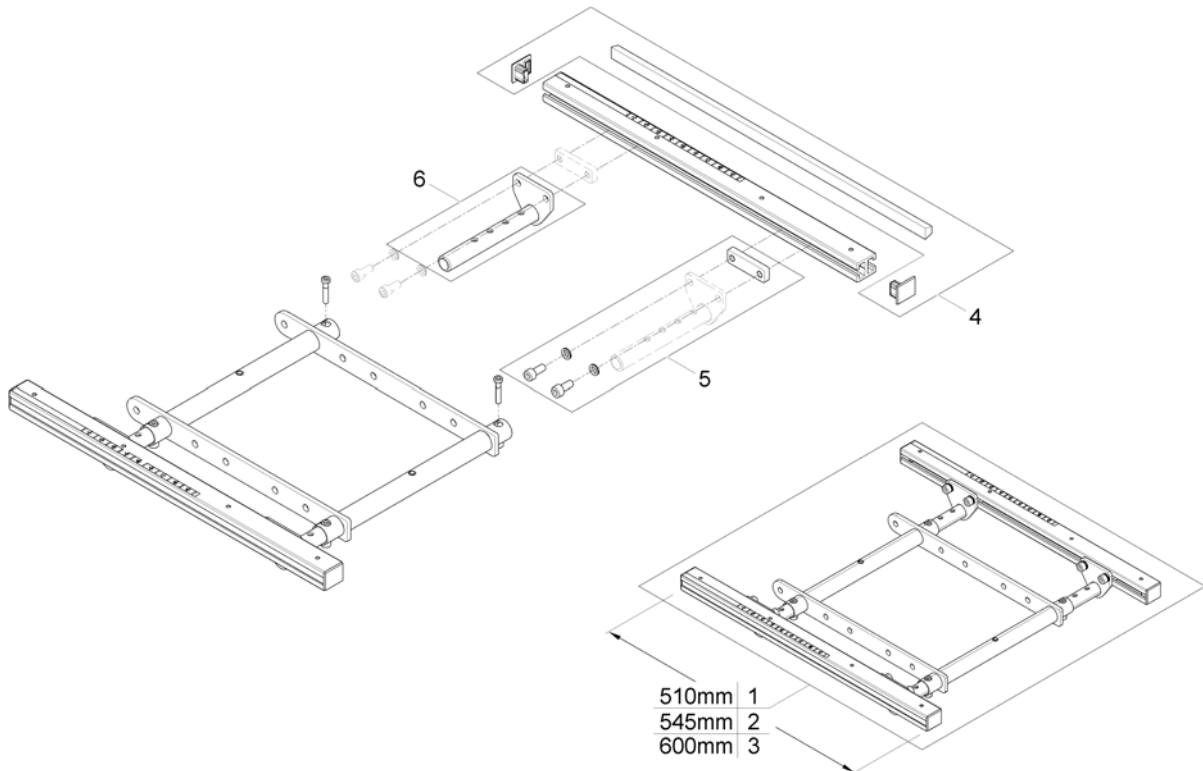

Pos.	Part number	Description	Valid from → until	Qty
-	SP1588443	Seat Rails 510mm (2pcs kit)		1
-	SP1588444	Seat Rails 545mm (2pcs kit)		1
-	SP1588445	Seat Rails 600mm (2pcs kit)		1
1	SP1572978	Seatframe 510mm, cpl., pre set up to SW 43cm		1
2	SP1582217	Seatframe 545mm, cpl., pre set up to SW 43cm		1
3	SP1582741	Seatframe 600mm, cpl., pre set up to SW 43cm		1
4	SP1571810	Sealing strap with plugs		1
5	SP1571811	Groove stone with mounting parts		1
6	SP1555400	Adjustment Tube		1

## Telescopic Seat Frame for Tilt 30° / Lifter

1582427.iso

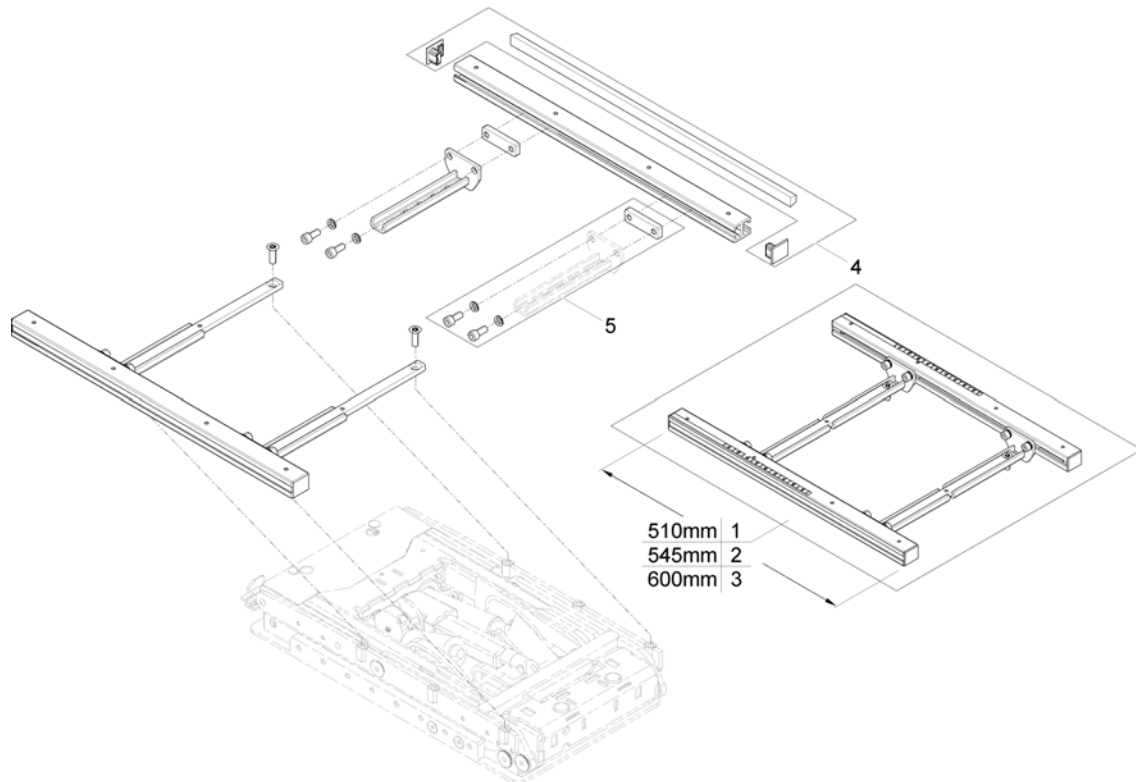


Pos.	Part number	Description	Valid from → until	Qty
-	SP1588443	Seat Rails 510mm (2pcs kit)		1
-	SP1588444	Seat Rails 545mm (2pcs kit)		1
-	SP1588445	Seat Rails 600mm (2pcs kit)		1
1	SP1572993	Seatframe 510mm, pre set up to SW 430mm, cpl.		1
2	SP1582230	Seatframe 545mm, pre set up to SW 430mm, cpl.		1
3	SP1582742	Seatframe 600mm, pre set up to SW 430mm, cpl.		1
4	SP1571810	Sealing strap with plugs		1
5	SP1571811	Groove stone with mounting parts		1



# Legrest Support

1572030.060

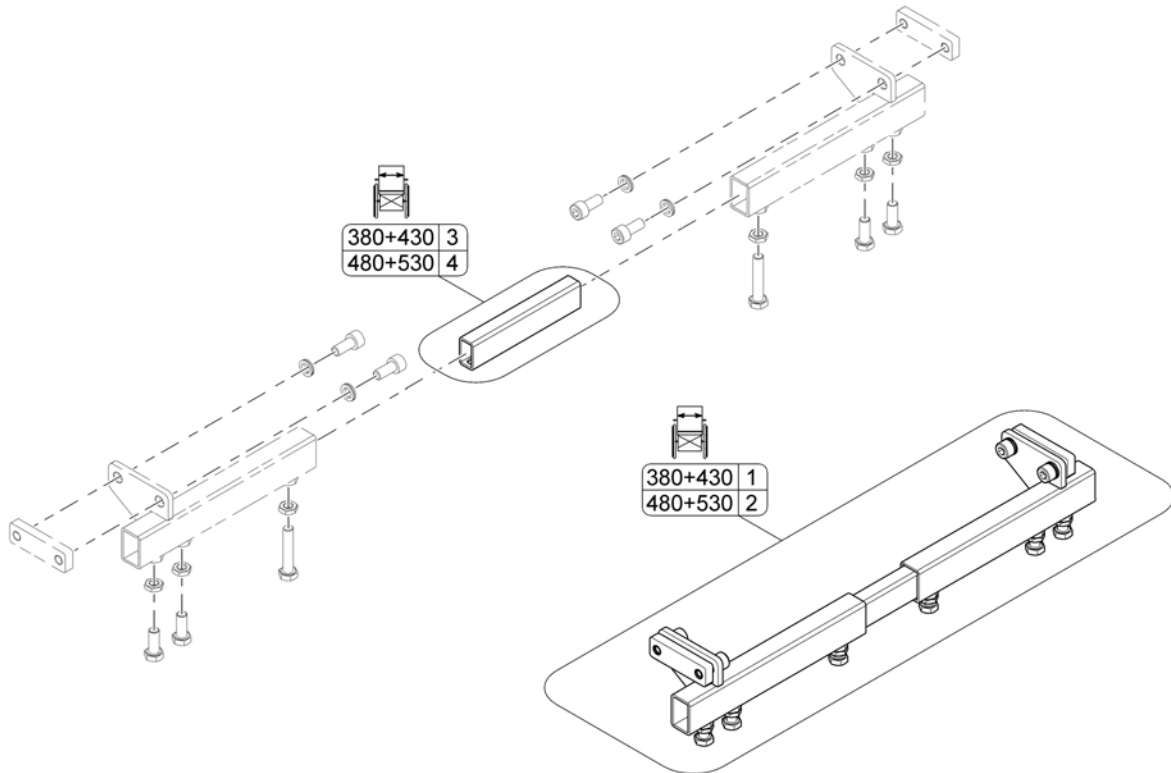


Pos.	Part number	Description	Valid from → until	Qty
1	SP1559305	Legrest Support SW380+430mm, cpl.		1
2	SP1559307	Legrest Support SW480+530mm, cpl.		1
3	SP1558134	Connection Tube SW380+430mm		1
4	SP1558135	Connection Tube SW480+530mm		1

# Sling Seat

1582429 iso


	SW	X
①	380/430	510mm
②	480/530	510mm
③	380/430	600mm
④	480/530	600mm

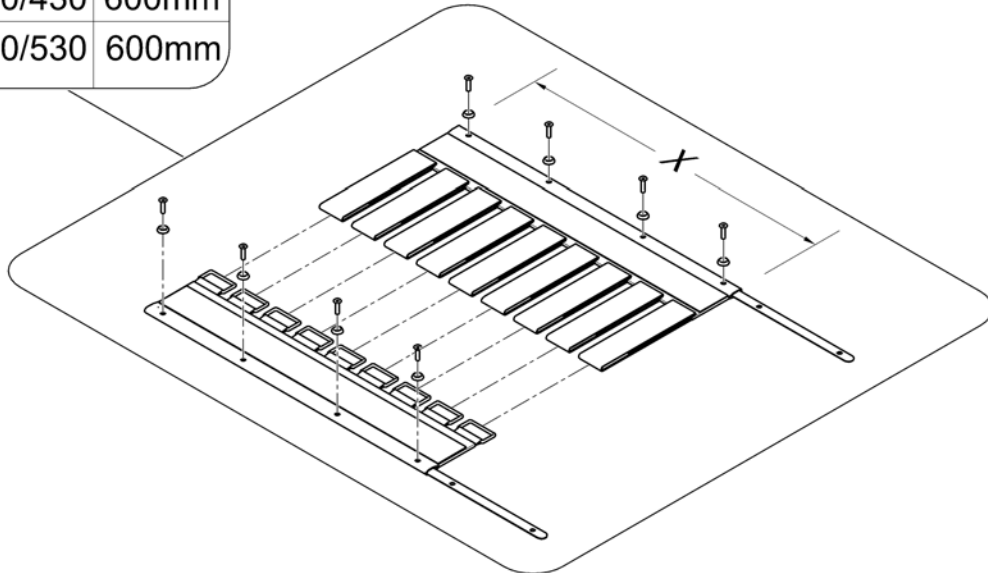

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557877	Sling Seat SW380/430, 510mm		1
2	SP1555911	Sling Seat SW480/530, 510mm		1
3	SP1580757	Sling Seat SW380/430, 600mm		1
4	SP1580758	Sling Seat SW480/530, 600mm		1

# Telescopic Seat Frame and Universal Adapter


1581864 iso

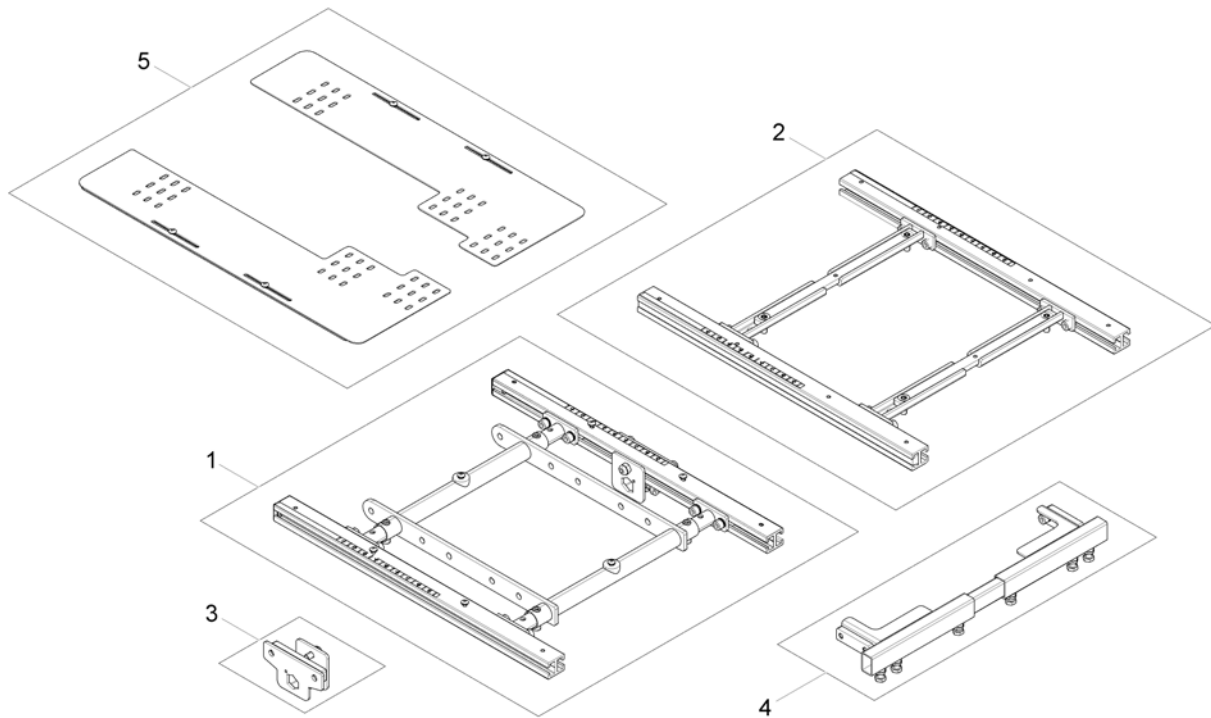

Pos.	Part number	Description	Valid from → until	Qty
1	-	Telescopic Seat Frame for man. /pow. 20° Tilt		1
2	-	Telescopic Seat Frame for Tilt 30° / Lifter		1
3	-	Armrest Bracket		1
4	-	Legrest Support		1
5	-	Adapter Kit for Universal Seat		1

# Adapter Kit for Universal Seat


1582430.iso

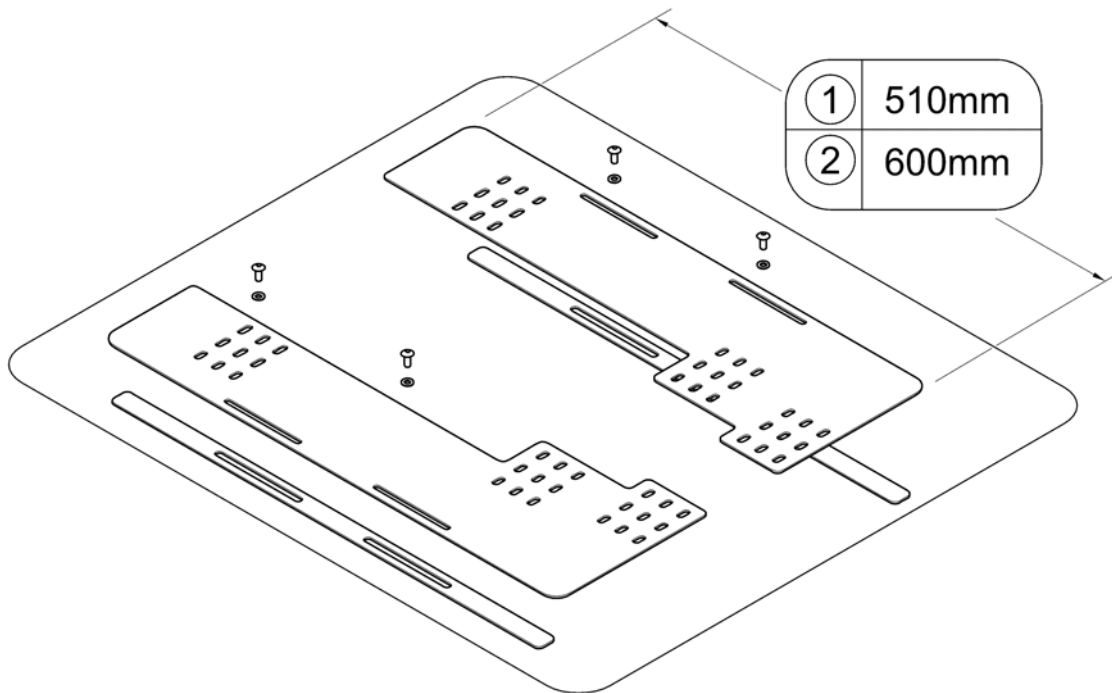

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580750	Adapter Kit for Universal Seat 510mm		1
2	SP1580759	Adapter Kit for Universal Seat 600mm		1

# Seat Cushion

1582643 iso

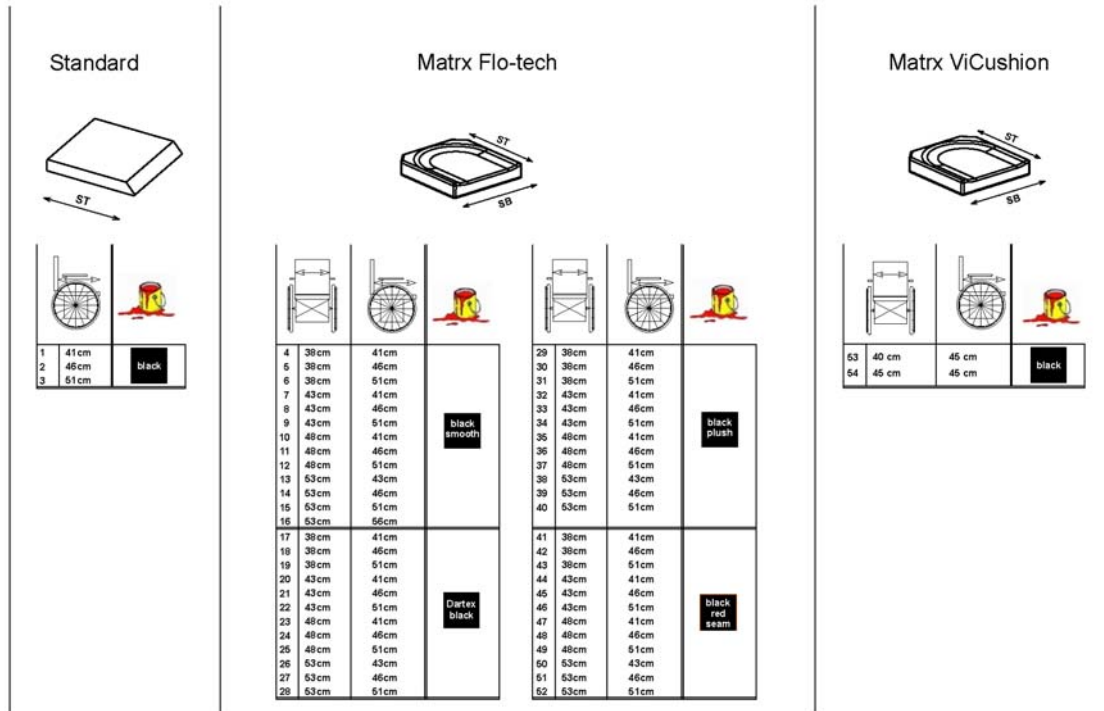

Pos.	Part number	Description	Valid from → until	Qty
1	1420460	Seat Cushion Standard SD 410mm <i>[2141]</i>		1
2	1420461	Seat Cushion Standard SD 460mm <i>[2141]</i>		1
3	1420462	Seat Cushion Standard SD 510mm <i>[2141]</i>		1
4	SPFLB15/16	Seatcushion Easy-Adapt SB38 x ST41cm black smooth <i>[2142+2151]</i>		1
5	SPFLB15/18	Seatcushion Easy-Adapt SB38 x ST46cm black smooth <i>[2142+2151]</i>		1
6	SPFLB15/20	Seatcushion Easy-Adapt SB38 x ST51cm black smooth <i>[2142+2151]</i>		1
7	SPFLB17/16	Seatcushion Easy-Adapt SB43 x ST41cm black smooth <i>[2142+2151]</i>		1
8	SPFLB17/18	Seatcushion Easy-Adapt SB43 x ST46cm black smooth <i>[2142+2151]</i>		1
9	SPFLB17/20	Seatcushion Easy-Adapt SB43 x ST51cm black smooth <i>[2142+2151]</i>		1
10	SPFLB19/16	Seatcushion Easy-Adapt SB48 x ST41cm black smooth <i>[2142+2151]</i>		1
11	SPFLB19/18	Seatcushion Easy-Adapt SB48 x ST46cm black smooth <i>[2142+2151]</i>		1
12	SPFLB19/20	Seatcushion Easy-Adapt SB48 x ST51cm black smooth <i>[2142+2151]</i>		1
13	SPFCB21/17	Seatcushion Easy-Adapt SB53 x ST43cm black smooth <i>[2142+2151]</i>		1
14	SPFCB21/18	Seatcushion Easy-Adapt SB53 x ST46cm black smooth <i>[2142+2151]</i>		1
15	SPFCB21/20	Seatcushion Easy-Adapt SB53 x ST51cm black smooth <i>[2142+2151]</i>		1
16	SP1578042	Seatcushion Flo-tech Contour SW53 x SD56cm black <i>[2142+2151]</i>		1
17	SP1529546-TR26	seat cushion Flo-tech Lite SW 38 x SD 41 Dartex black <i>[2142+2152]</i>		1
18	SP1529547-TR26	seat cushion Flo-tech Lite SW 38 x SD 46 Dartex black <i>[2142+2152]</i>		1

## Seat Cushion


Pos.	Part number	Description	Valid from → until	Qty
19	SP1529548-TR26	seat cushion Flo-tech Lite SW 38 x SD 51 Dartex black <i>[2142+2152]</i>		1
20	SP1529549-TR26	seat cushion Flo-tech Lite SW 43 x SD 41 Dartex black <i>[2142+2152]</i>		1
21	SP1529550-TR26	seat cushion Flo-tech Lite SW 43 x SD 46 Dartex black <i>[2142+2152]</i>		1
22	SP1529551-TR26	seat cushion Flo-tech Lite SW 43 x SD 51 Dartex black <i>[2142+2152]</i>		1
23	SP1529552-TR26	seat cushion Flo-tech Lite SW 48 x SD 41 Dartex black <i>[2142+2152]</i>		1
24	SP1529553-TR26	seat cushion Flo-tech Lite SW 48 x SD 46 Dartex black <i>[2142+2152]</i>		1
25	SP1529554-TR26	seat cushion Flo-tech Lite SW 48 x SD 51 Dartex black <i>[2142+2152]</i>		1
26	SP1529555-TR26	seat cushion Flo-tech Contour SW 53 x SD 43 Dartex black <i>[2142+2152]</i>		1
27	SP1529556-TR26	seat cushion Flo-tech Contour SW 53 x SD 46 Dartex black <i>[2142+2152]</i>		1
28	SP1529570-TR26	seat cushion Flo-tech Contour SW 53 x SD 51 Dartex black <i>[2142+2152]</i>		1
29	SP1529546-TR35	seat cushion Flo-tech Lite SW 38 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
30	SP1529547-TR35	seat cushion Flo-tech Lite SW 38 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
31	SP1529548-TR35	seat cushion Flo-tech Lite SW 38 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
32	SP1529549-TR35	seat cushion Flo-tech Lite SW 43 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
33	SP1529550-TR35	seat cushion Flo-tech Lite SW 43 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
34	SP1529551-TR35	seat cushion Flo-tech Lite SW 43 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
35	SP1529552-TR35	Seat cushion Flo-tech Lite SW 48 x SD 41 black chenille fabric <i>[2142+2154]</i>		1
36	SP1529553-TR35	seat cushion Flo-tech Lite SW 48 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
37	SP1529554-TR35	seat cushion Flo-tech Lite SW 48 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
38	SP1529555-TR35	seat cushion Flo-tech Contour SW 53 X SD 43 black chenille fabric <i>[2142+2154]</i>		1
39	SP1529556-TR35	seat cushion Flo-tech Contour SW 53 x SD 46 black chenille fabric <i>[2142+2154]</i>		1
40	SP1529570-TR35	seat cushion Flo-tech Contour SW 53 x SD 51 black chenille fabric <i>[2142+2154]</i>		1
41	SP1529546	Seatcushion Easy-Adapt SB38 x ST41cm black with red seam <i>[2142+2153]</i>		1
42	SP1529547	Seatcushion Easy-Adapt SB38 x ST46cm black with red seam <i>[2142+2153]</i>		1
43	SP1529548	Seatcushion Easy-Adapt SB38 x ST51cm black with red seam <i>[2142+2153]</i>		1
44	SP1529549	Seatcushion Easy-Adapt SB43 x ST41cm black with red seam <i>[2142+2153]</i>		1
45	SP1529550	Seatcushion Easy-Adapt SB43 x ST46cm black with red seam <i>[2142+2153]</i>		1
46	SP1529551	Seatcushion Easy-Adapt SB43 x ST51cm black with red seam <i>[2142+2153]</i>		1
47	SP1529552	Seatcushion Easy-Adapt SB48 x ST41cm black with red seam <i>[2142+2153]</i>		1
48	SP1529553	Seatcushion Easy-Adapt SB48 x ST46cm black with red seam <i>[2142+2153]</i>		1
49	SP1529554	Seatcushion Easy-Adapt SB48 x ST51cm black with red seam <i>[2142+2153]</i>		1
50	SP1529555	Seatcushion Easy-Adapt SB53 x ST43cm black with red seam <i>[2142+2153]</i>		1

**Seat Cushion**

Pos.	Part number	Description	Valid from → until	Qty
51	SP1529556	Seatcushion Easy-Adapt SB53 x ST46cm black with red seam <i>[2142+2153]</i>		1
52	SP1529570	Seatcushion Easy-Adapt SB53 x ST51cm black with red seam <i>[2142+2153]</i>		1
53	SP1531760	Seat Cushion MatrX Vi SW 40 x SD 45cm (16" x 18") <i>[2144]</i>		1
54	SP1531761	Seat Cushion MatrX Vi SW 45 x SD 45cm (18" x 18") <i>[2144]</i>		1
55	SP1501113	Seat Cushion Flo-tech Contur Visco 43x43cm <i>[2143]</i>		1

# Max Seat


1582428.iso

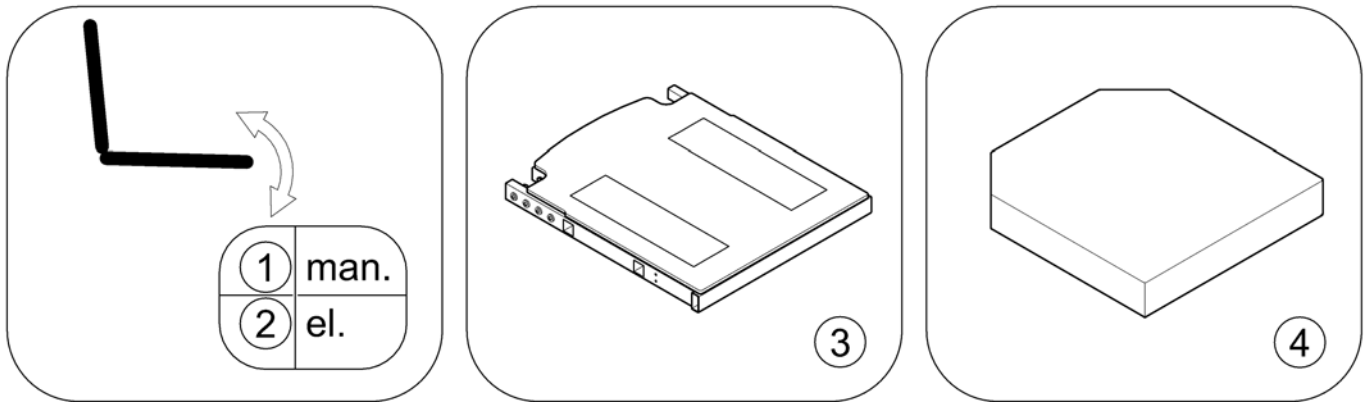

Pos.	Part number	Description	Valid from → until	Qty
1	-	man. settable Tilt		1
2	-	powered simple Tilt		1
3	-	Seat Frame and Plate		1
4	-	Seat Cushion		1




1582059-00

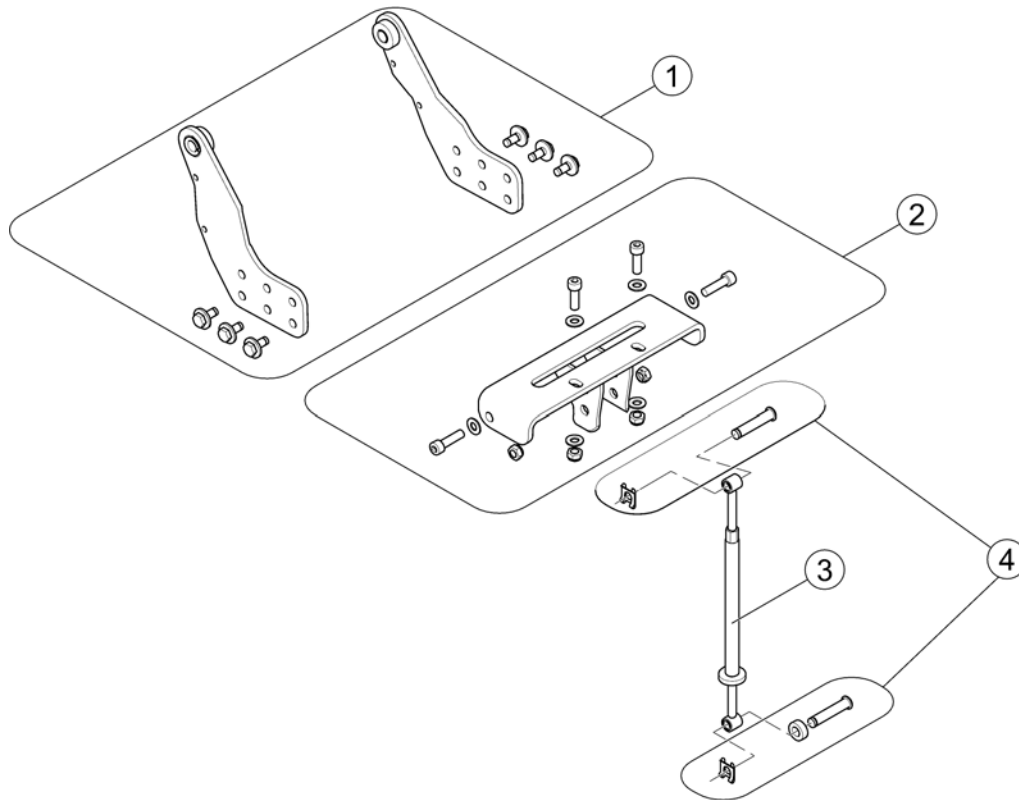

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582746	Pivot Bearing Back - Max Seat		1
2	SP1582747	Support Plate Tilt - Max Seat		1
3	1420003	SPINDLE SEAT HEIGHT ADJUSTMENT MANUAL CPL.		1
4	SP1582737	Mounting Hardware for Tilt		1


# powered simple Tilt

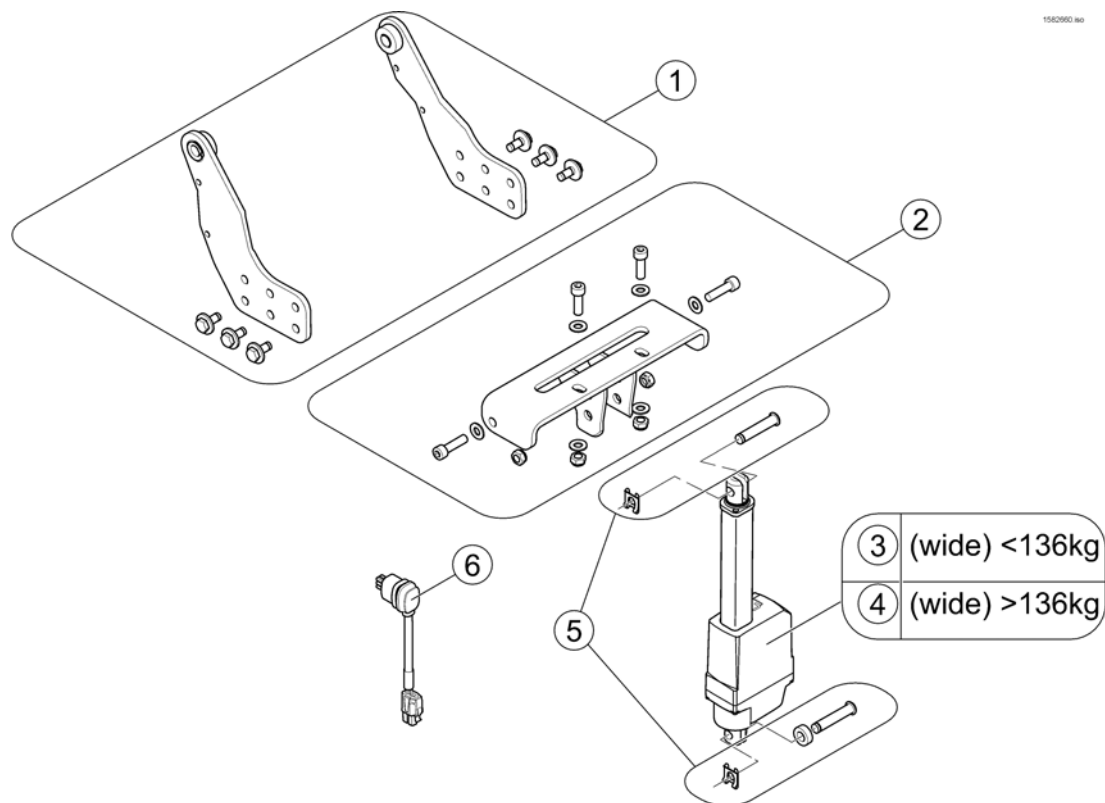

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582746	Pivot Bearing Back - Max Seat		1
2	SP1582747	Support Plate Tilt - Max Seat		1
3	SP1581020	Actuator LA23 - stroke 135mm / 1800N		1
4	SP1556487	Actuator LA23 - stroke 135mm / 2500N		1
5	SP1582737	Mounting Hardware for Tilt		1
6	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1

## Seat Frame and Plate


1582061.000

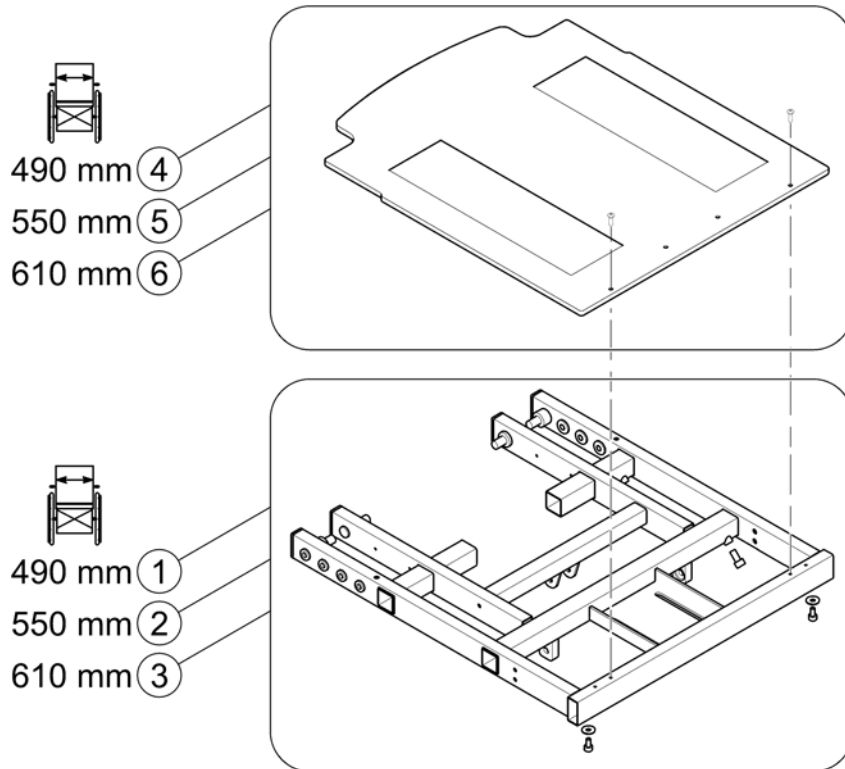

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582748	Seat Frame SW 490mm		1
2	SP1582749	Seat Frame SW 550mm		1
3	SP1582772	Seat Frame SW 610mm		1
4	SP1508137	Seat plate SW 490mm		1
5	SP1525083	Seat plate SW 550mm		1
6	SP1525082	Seat plate SW 610mm		1

# Seat Cushion


1582666.iso

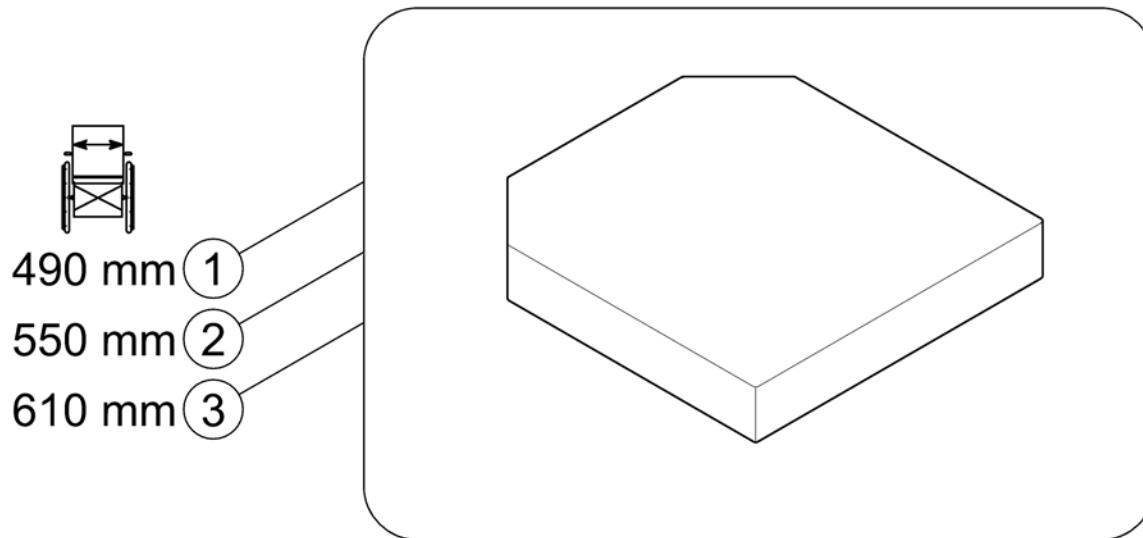

Pos.	Part number	Description	Valid from → until	Qty
1	SP1508059	Seat cushion 505x535, TR23		1
2	SP1536595	Seat cushion 550 x 635, TR23		1
3	SP1536596	Seat cushion 610x635, TR23		1

# Backrest


SPCTDX\_SP220.iso

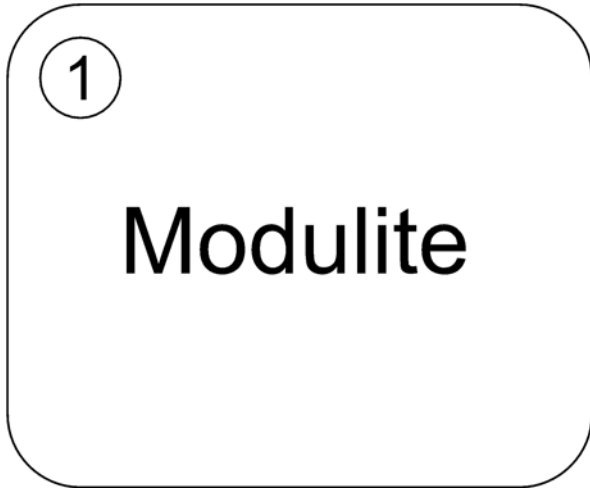

Pos.	Part number	Description	Valid from → until	Qty
1	-	Backrest for Modulite Seat		1
2	-	Backrest for Max Seat		1


# Backrest for Modulite Seat

1572325-00

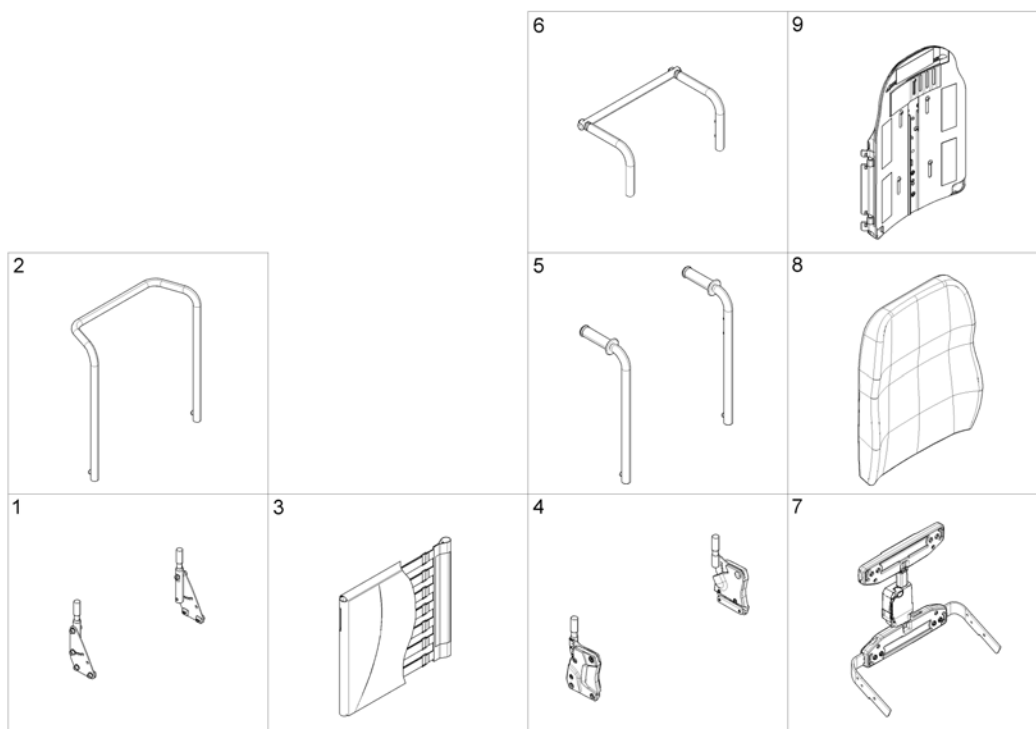

Pos.	Part number	Description	Valid from → until	Qty
1	-	Support for Backrest Frame <i>[0975+2161]</i>		1
2	-	Backrest Frame <i>[2161]</i>		1
3	-	Sling Back, tension adjustable <i>[2171]</i>		1
4	-	Backrest support		1
5	-	Backposts <i>[2162/2163]</i>		1
6	-	Connection tube		1
7	-	Back Recliner / Stabilization bracket <i>[0977]</i>		1
8	-	Backrest Cushion		1
9	-	Flex3 adjustable Backrest Plate <i>[2172/2173]</i>		1


1571553 iso

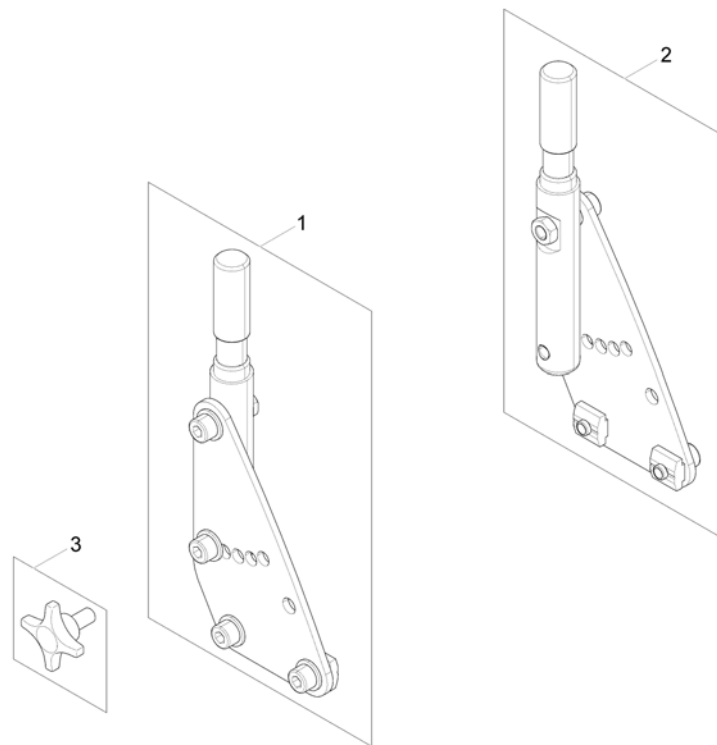

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571542	Backrest Support RH		1
2	SP1571553	Backrest Support LH		1
3	5824117	hand wheel M8x15 <i>(only swedish homologation)</i>		2

# Backrest Frame

1571831.00

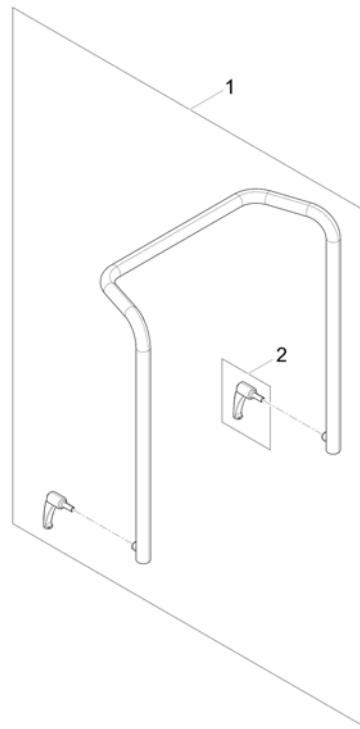


Pos.	Part number	Description	Valid from → until	Qty
1	SP1567810	Backrest Frame		1
2	0021-70002	Indexable lever 63mm M8x16		1



Invacare® TDX-SP2, TDX-SP2 NB | Backrest  
**Sling Back, tension adjustable**


1571829.000

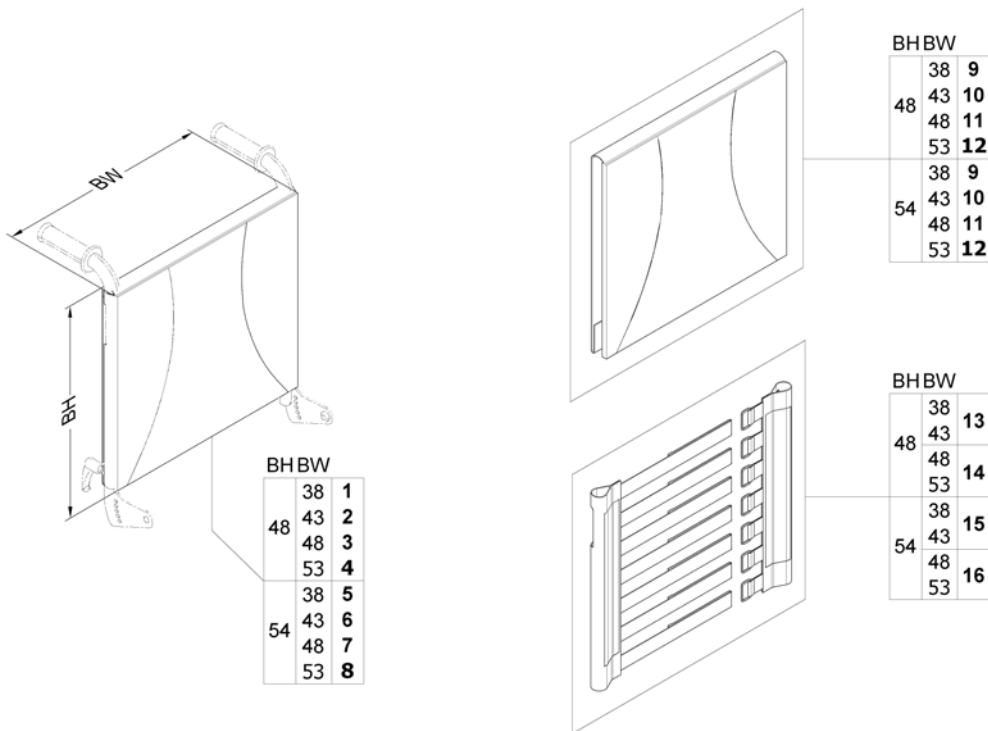

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567529	Sling Back, tension adjustable & Upholstery, BW 380mm / BH 480mm, cpl.		1
2	SP1567465	Sling Back, tension adjustable & upholstery, BW 430mm / BH 480mm, cpl.		1
3	SP1567530	Sling Back, tension adjustable & upholstery, BW 480mm / BH 480mm, cpl.		1
4	SP1567531	Sling Back, tension adjustable & upholstery, BW 530mm / BH 480mm, cpl.		1
5	SP1567627	Sling Back, tension adjustable & upholstery, BW 380mm / BH 540mm, cpl.		1
6	SP1567746	Sling Back, tension adjustable & upholstery, BW 430mm / BH 540mm, cpl.		1
7	SP1567747	Sling Back, tension adjustable & upholstery, BW 480mm / BH 540mm, cpl.		1
8	SP1567748	Sling Back, tension adjustable & upholstery, BW 530mm / BH 540mm, cpl.		1
9	SP1563563	Backrest upholstery		1
10	SP1563653	Backrest upholstery		1
11	SP1563654	Backrest upholstery		1
12	SP1563655	Backrest upholstery		1
13	SP1571492	Sling back		1
14	SP1571493	Sling back		1
15	SP1571494	Sling back		1
16	SP1571510	Sling back		1


# Backrest support

1572928 Rev

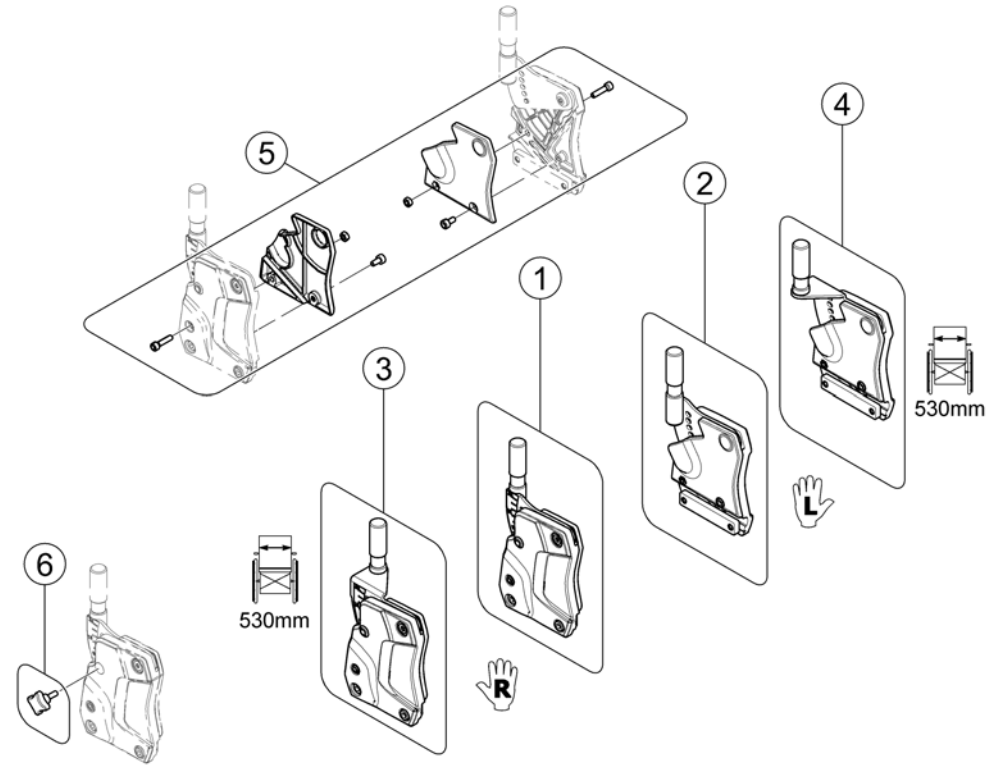

Pos.	Part number	Description	Valid from → until	Qty
1	SP1582709	Backrest Support - RH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1571560</i>		1
2	SP1582708	Backrest Support - LH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1571561</i>		1
3	SP1582952	Backrest Support SW 530mm - RH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1571568</i>		1
4	SP1582951	Backrest Support SW 530mm - LH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1571569</i>		1
5	SP1571676	Cover		1
6	SP1579877	Hand Screw M6 <i>(Swedish Homologation only!)</i>		2


# Backposts

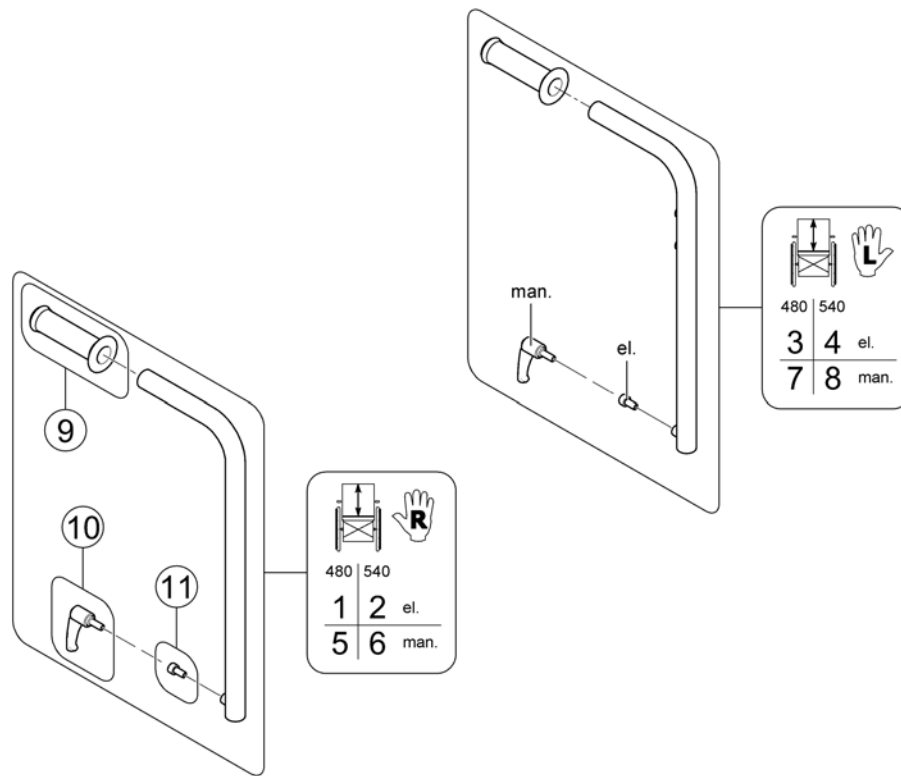

Pos.	Part number	Description	Valid from → until	Qty
1	SP1587177	Backpost el. BH 480mm - RH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1563102</i>		1
2	SP1587184	Backpost el. BH 540mm - RH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1563103</i>		1
3	SP1587176	Backpost el. BH 480mm - LH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1560576</i>		1
4	SP1587183	Backpost el. BH 540mm - LH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1559578</i>		1
5	SP1587163	Backpost man. BH 480mm - RH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1572489</i>		1
6	SP1587182	Backpost man. BH 540mm - RH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1572490</i>		1
7	SP1587162	Backpost man. BH 480mm - LH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1572491</i>		1
8	SP1587178	Backpost man. BH 540mm - LH <i>Compatible with Flip-Up &amp; Following Armrest [2193/2192/2194] - replaces SP1572492</i>		1
9	SP1563651	Handles (pair)		1
10	0021-70002	Indexable lever 63mm M8x16		1
11	1567999	Bolt M8x16 8.8 black DIN912		1

## Connection tube


1571835 no

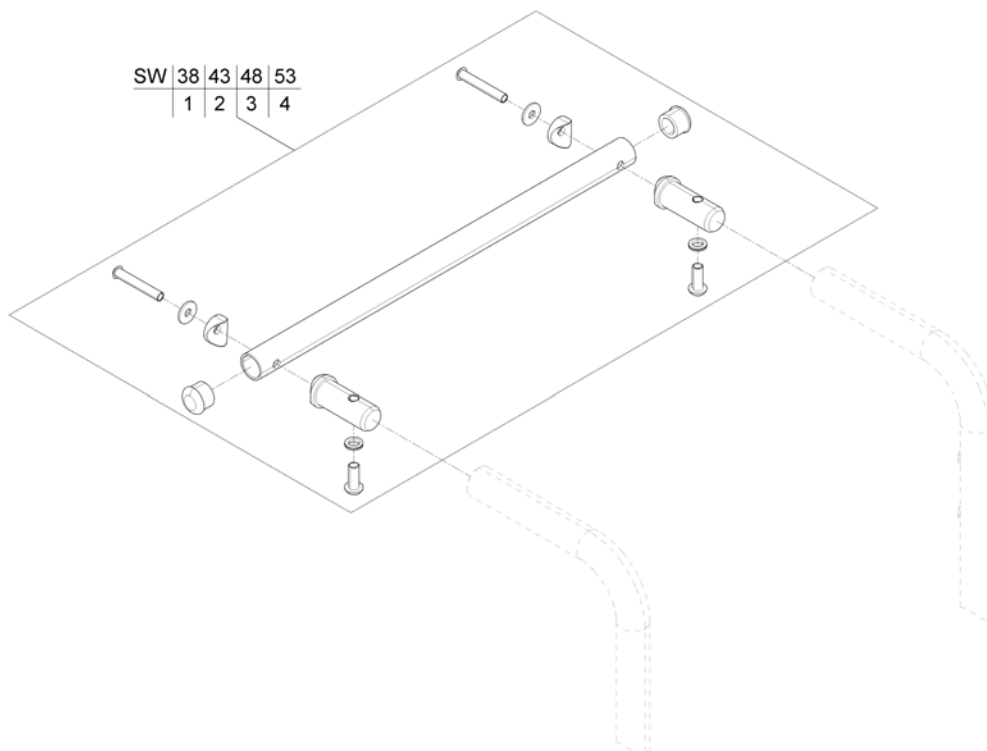

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570370	Connection tube		1
2	SP1570371	Connection tube		1
3	SP1570372	Connection tube		1
4	SP1570373	Connection tube		1

## Back Recliner / Stabilization bracket


1562743.kso

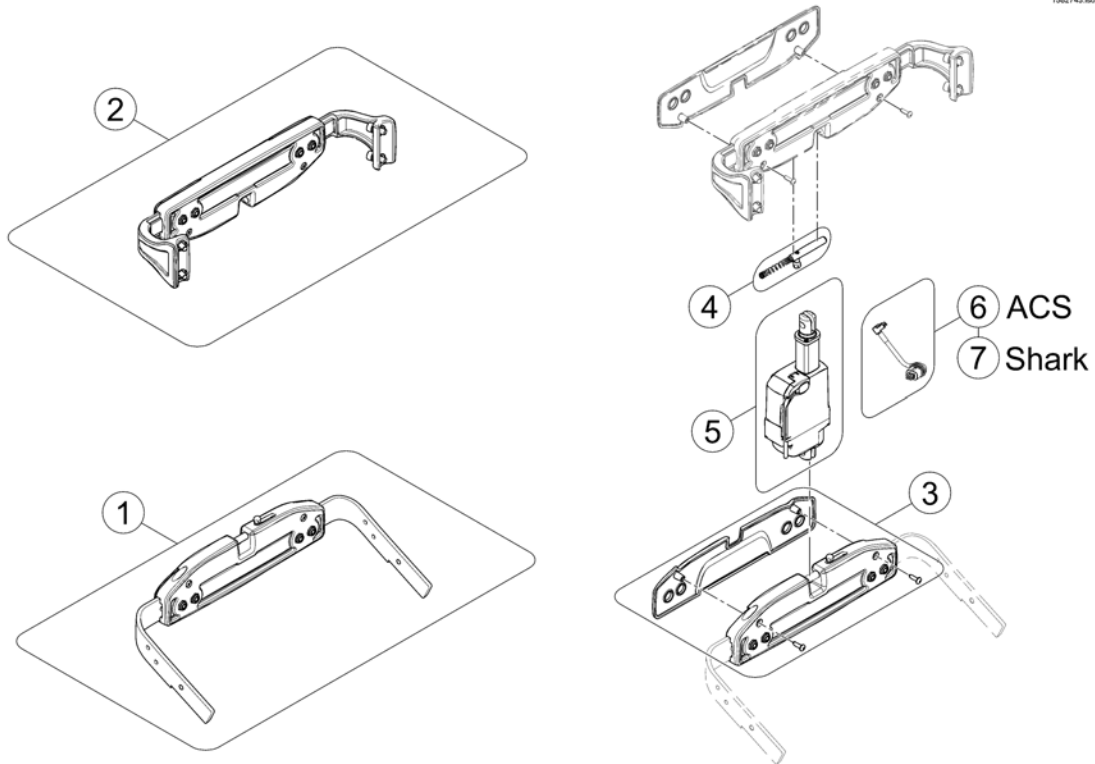

Pos.	Part number	Description	Valid from → until	Qty
1	SP1567169	Actuator bracket (pre set to SW 43cm)		1
2	SP1567168	Stabilization bracket (pre set to SW 43cm)		1
3	SP1571702	Cover		1
4	SP1571704	Locking bolt		1
5	SP1557820	Actuator		1
6	SP1572166	Actuator Cable LA23 ACS2 (~950mm)		1
7	SP1552297	Actuator Cable LA23 Shark (~500mm)		1

# Backrest Cushion

1573106.no

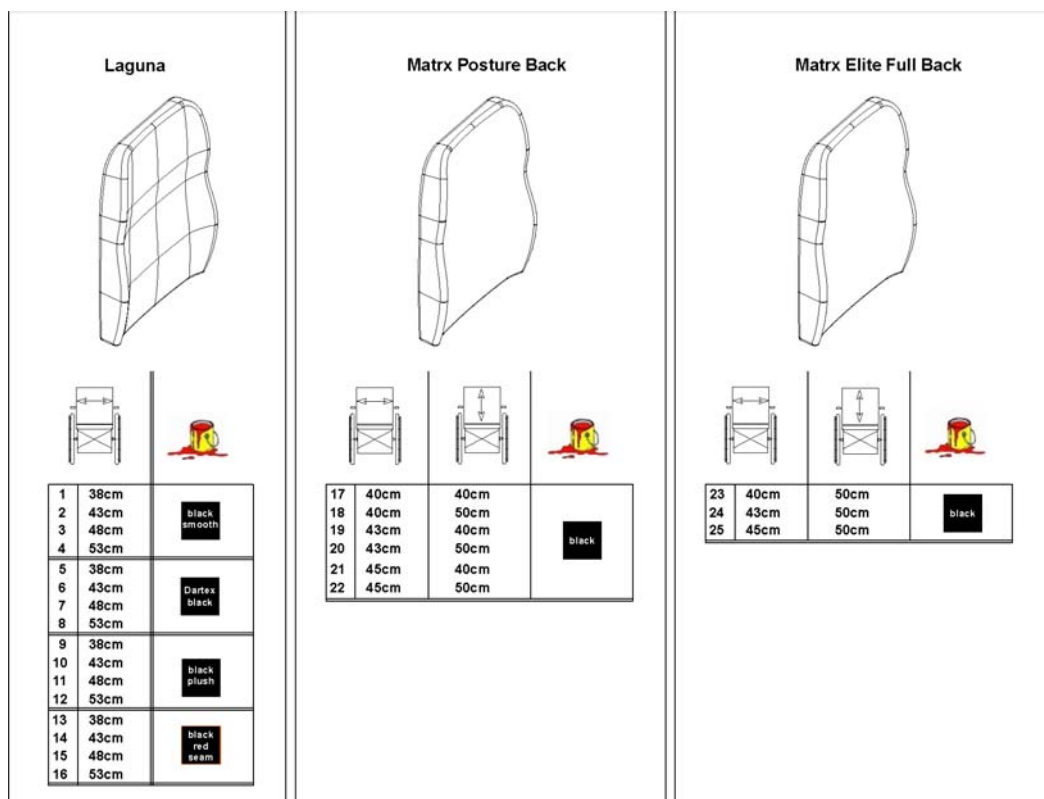


Pos.	Part number	Description	Valid from → until	Qty
1	SP1529585	Backrestcushion Laguna shape, Black, 38cm <i>[2173+2151]</i>		1
2	SP1529586	Backrestcushion Laguna shape, Black, 43cm <i>[2173+2151]</i>		1
3	SP1529587	Backrestcushion Laguna shape, Black, 48cm <i>[2173+2151]</i>		1
4	SP1529596	Backrestcushion Laguna shape, Black, 53cm <i>[2173+2151]</i>		1
5	SP1529538-TR26	Backrest cushion Laguna SW 38 Black Dartex <i>[2173+2152]</i>		1
6	SP1529539-TR26	Backrest cushion Laguna SW 43 Black Dartex <i>[2173+2152]</i>		1
7	SP1529540-TR26	Backrest cushion Laguna SW 48 Black Dartex <i>[2173+2152]</i>		1
8	SP1529545-TR26	Backrest cushion Laguna SW 53 Black Dartex <i>[2173+2152]</i>		1
9	SP1529538-TR35	Backrest cushion Laguna SW 38 Black Plush <i>[2173+2154]</i>		1
10	SP1529539-TR35	Backrest cushion Laguna SW 43 Black Plush <i>[2173+2154]</i>		1
11	SP1529540-TR35	Backrest cushion Laguna SW 48 Black Plush <i>[2173+2154]</i>		1
12	SP1529545-TR35	Backrest cushion Laguna SW 53 Black Plush <i>[2173+2154]</i>		1
13	SP1529538	Backrestcushion Laguna shape, Black with orange seam, 38cm <i>[2173+2153]</i>		1
14	SP1529539	Backrestcushion Laguna shape, Black with orange seam, 43cm <i>[2173+2153]</i>		1
15	SP1529540	Backrestcushion Laguna shape, Black with orange seam, 48cm <i>[2173+2153]</i>		1
16	SP1529545	Backrestcushion Laguna shape, Black with orange seam, 53cm <i>[2173+2153]</i>		1
17	SP1531762	Matrix "Posture Back" Back 40x40cm (16" x 16") <i>[2174]</i>		1
18	SP1531763	Matrix "Posture Back" Back 40x50cm (16" x 20") <i>[2175]</i>		1

**Backrest Cushion**

<b>Pos.</b>	<b>Part number</b>	<b>Description</b>	<b>Valid from → until</b>	<b>Qty</b>
19	SP1531792	Matrx "Posture Back" Back 43x40cm (17" x 16") [2174]		1
20	SP1531793	Matrx "Posture Back" Back 43x50cm (17" x 20") [2175]		1
21	SP1531822	Matrx "Posture Back" Back 45x40cm (18" x 16") [2174]		1
22	SP1531823	Matrx "Posture Back" Back 45x50cm (18" x 20") [2175]		1
23	SP1531824	Matrx "Elite Full Back" Back 40x50cm (16" x 20") [2176]		1
24	SP1531831	Matrx "Elite Full Back" Back 43x50cm (17" x 20") [2176]		1
25	SP1531832	Matrx "Elite Full Back" Back 45x50cm (18" x 20") [2176]		1

## Flex3 adjustable Backrest Plate

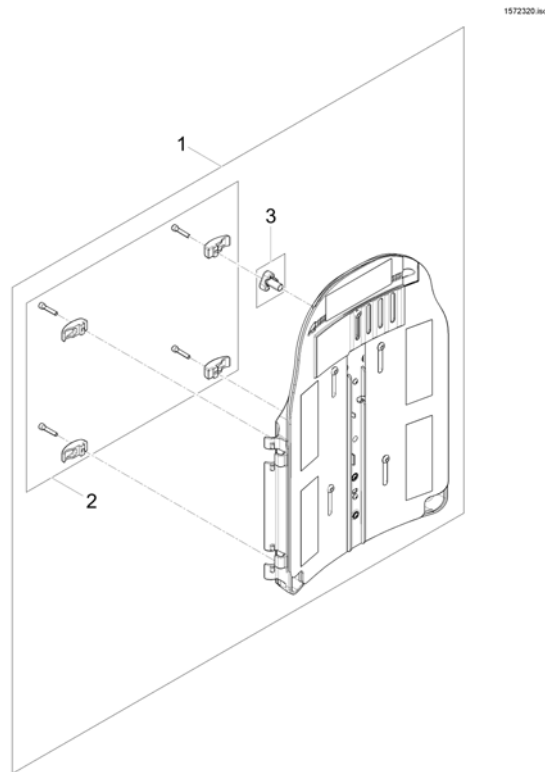


Pos.	Part number	Description	Valid from → until	Qty
1	SP1568489	Flex3 backrest plate		1
2	SP1572321	Clamps		1
3	1497411	HANDWHEEL M12 LONG NECK/HEADREST		1



# Backrest for Max Seat



1582669.iso

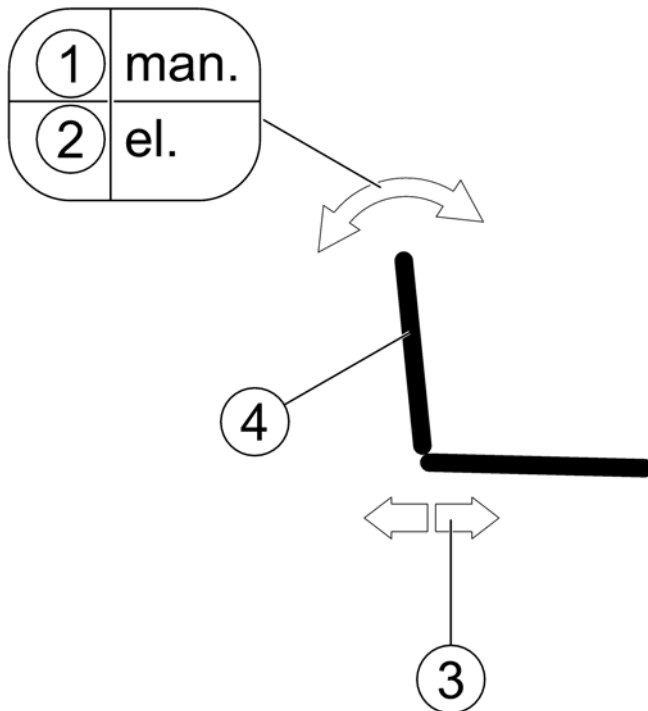


Pos.	Part number	Description	Valid from → until	Qty
1	-	Manual backrest adjustment		1
2	-	Powered backrest adjustment		1
3	-	Seat depth adapter		1
4	-	Backrest		1

# Manual backrest adjustment

1545185.jpg

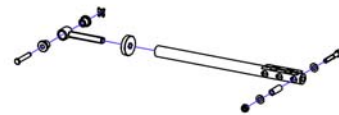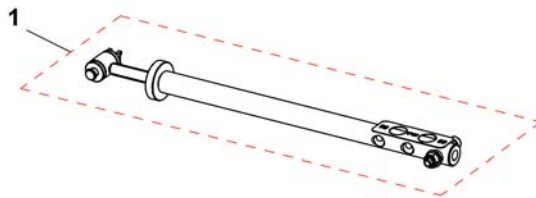


Pos.	Part number	Description	Valid from → until	Qty
1	SP1545185	Spindle, backrest adjustment, cpl.		1

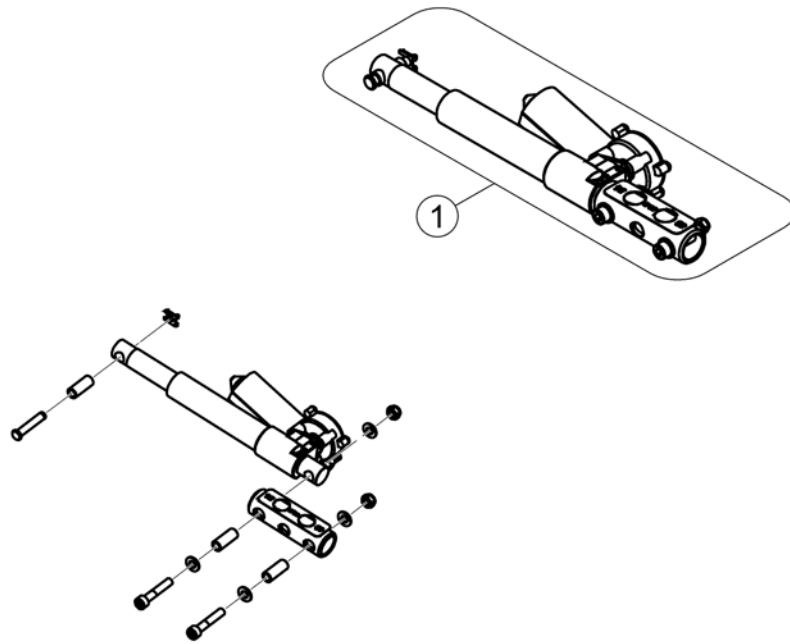


Pos.	Part number	Description	Valid from → until	Qty
1	SP1544669	Actuator, cpl.		1

# Seat depth adapter

1551724-01

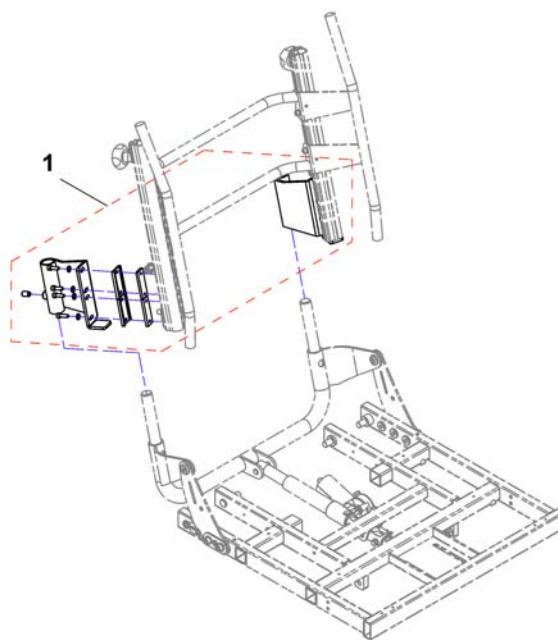



Pos.	Part number	Description	Valid from → until	Qty
1	SP1551724	Seat depth adapter, cpl.		1

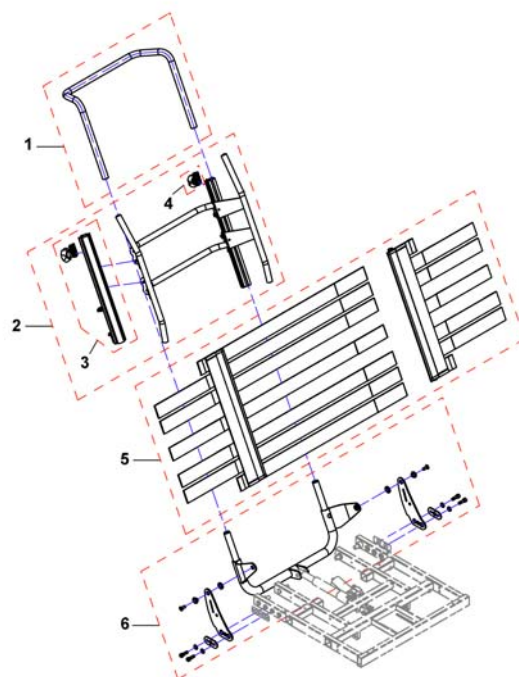**Backrest**

Pos.	Part number	Description	Valid from → until	Qty
1	SP1543082	Support tube, SW49		1
1	SP1543081	Support tube, SW55		1
1	SP1543080	Support tube, SW61		1
2	SP1551199	Backrest frame, cpl., SW49		1
2	SP1526824	Backrest frame, cpl., SW55		1
2	SP1526823	Backrest frame, cpl., SW61		1
3	1445848	Backrest Rail, cpl.		1
4	5253088	Fixation block		1
-	1508867	Cover SW49, dartex grey <i>not shown</i>		1
-	SP1526459	Cover, Lateral 640X520, Dartex Grey <i>not shown</i>		1
-	1526424	Cover, Lateral 700X500, Dartex Grey <i>not shown</i>		1
5	1445694	Backrest, tension adjustable, SW up to 49cm		1
5	SP1526308	Backrest, tension adjustable, SW 55 - 61cm		1
6	SP1544310	Backrest bracket, cpl., SW49		1
6	SP1544311	Backrest bracket, cpl., SW55		1
6	SP1544312	Backrest bracket, cpl., SW61		1

# Legrests

SPCTDX\_SP210.iso

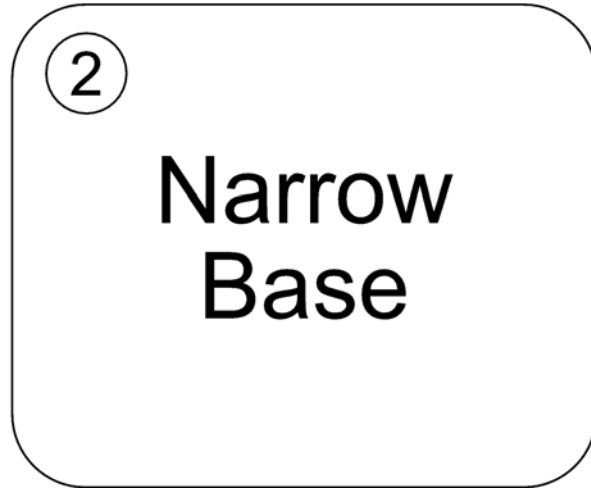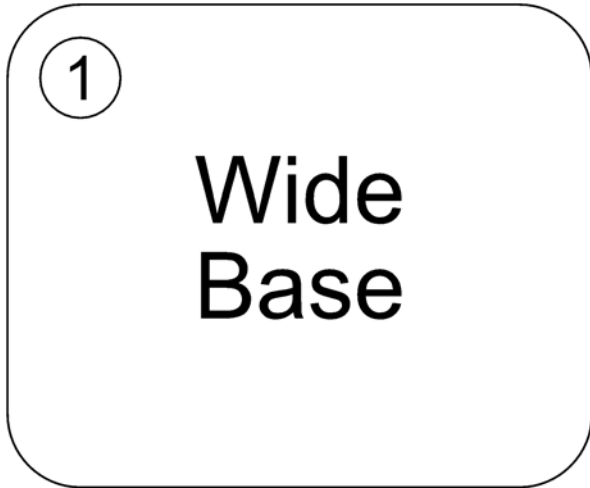

Pos.	Part number	Description	Valid from → until	Qty
1	-	Legrests - Wide Base		0
2	-	Legrests - Narrow Base		0

## Legrests - Wide Base

Pos.	Part number	Description	Valid from → until	Qty
1	-	Modulite Seat		0
2	-	Max Seat		0


## Modulite Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Legrest Support 5cm, Vari series		0
-		Hangers Vari A LH/RH 04.2009 =>		0
-		Hangers Vari F LH/RH		0
-		Hangers ADM LH/RH manual		0
-		Hangers ADE/ADP LH/RH electric 07.2005 =>		0
-		Footrests Vari, Flip-Up - 01.08.10 =>		0
-		Footrests Vari, Flip-Up & angle adjustable - 01.08.10 =>		0
-		Footrests Vari, angle & depth adjustable		0
-		Footboard Vari, pre-assembled <i>Football Chair only!</i>		0
-		Support Centr. Legrest LNX man. - Modulite Seat		0
-		Support Centr. Legrest LNX el. - Modulite Seat		0
-		Centr. Legrest man.		0
-		Centr. Legrest LNX el.		0




Invacare® TDX-SP2, TDX-SP2 NB | Legrests  
**Legrest Support 5cm, Vari series**

1573651.000

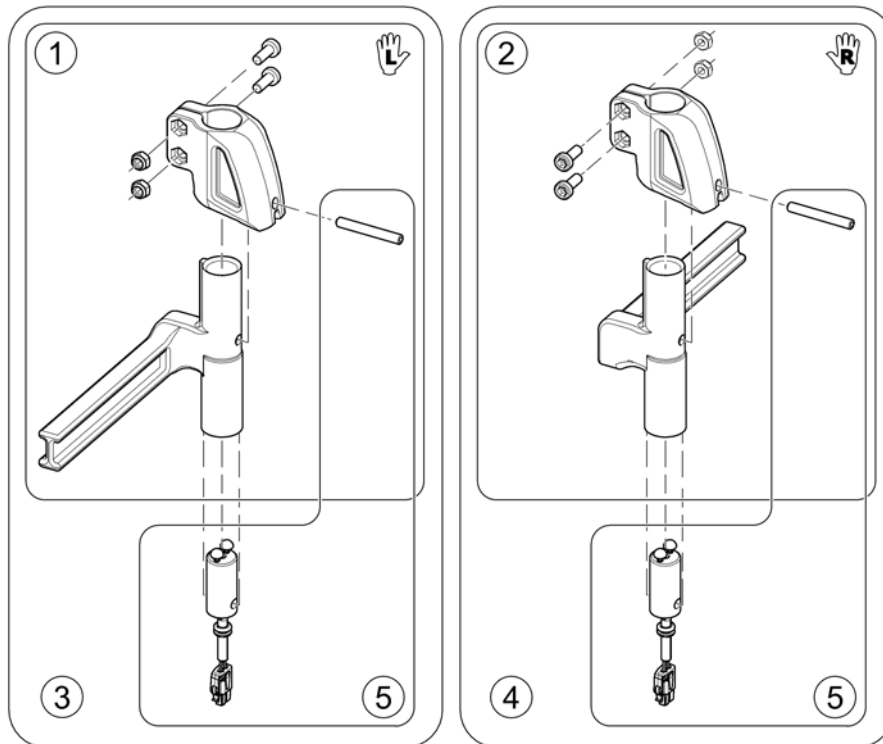

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449733	Footrest/Legrest receptacle, manuell, LH, 5cm		1
2	SP1449732	Footrest/Legrest receptacle, manuell, RH, 5cm		1
3	SP1529951	Footrest/Legrest receptacle, powered, LH, 5cm		1
4	SP1529949	Footrest/Legrest receptacle, powered, RH, 5cm		1
5	SP1527775	Power Supply Cable Legrest (650mm)		1


## Hangers Vari A LH/RH 04.2009 =>

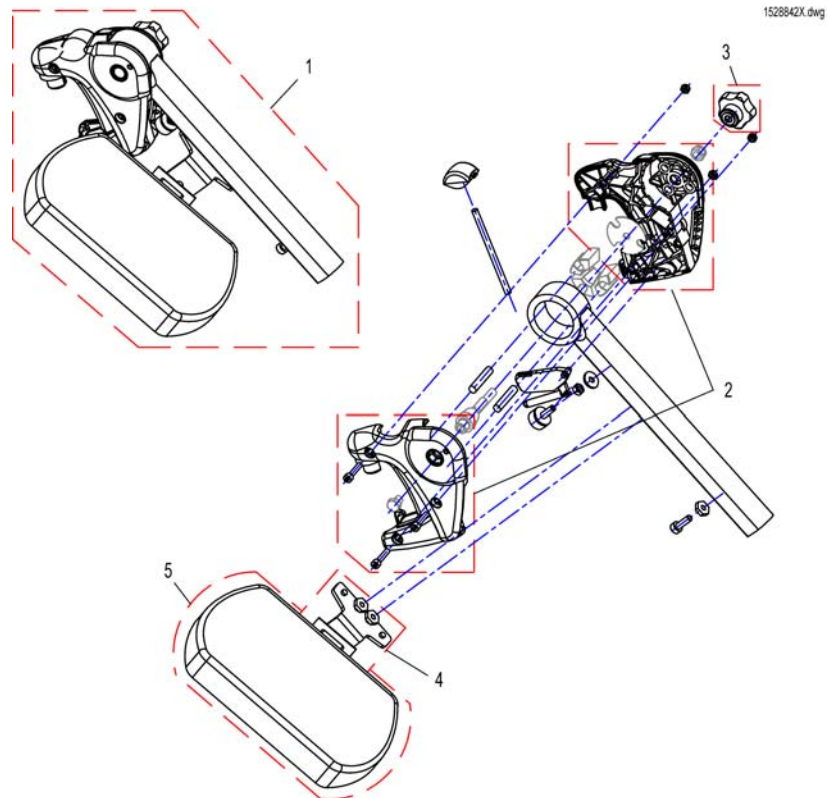

Pos.	Part number	Description	Valid from → until	Qty
-	SP1527820	Rework-Kit Vari A Legrest Version 1 => Version 2 <i>Vari A Version 1 =&gt; Version 2 Upgrade Kit</i>		1
1	SP1527498	Hanger Vari A LH, cpl.		1
1	SP1527492	Hanger Vari A RH, cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	L34011	Handknob M8 Tristarfemale		1
4	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
4	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
5	6301333	Legrest calf pad		1

# Hangers Vari F LH/RH


144934 / X.dwg

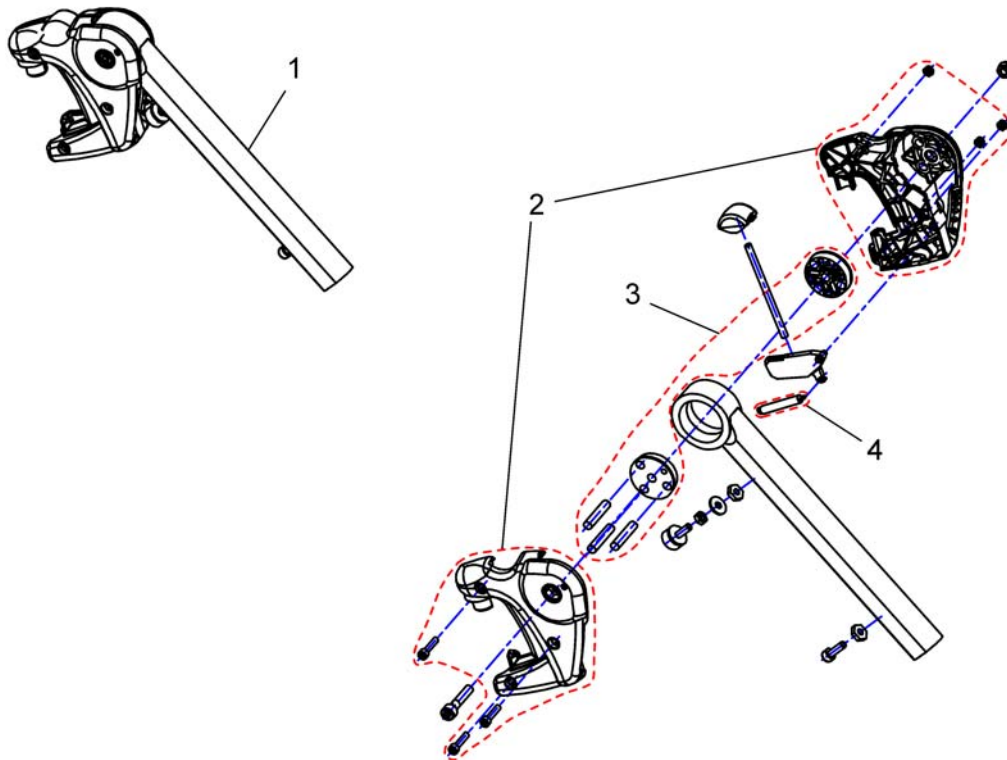

Pos.	Part number	Description	Valid from → until	Qty
1	SP1449347	Hanger Vari F LH cpl.		1
1	SP1449346	Hanger Vari F RH cpl.		1
2	SP1434955	Cover Legrest cpl.		1
3	1434956	Mechanic		1
4	SPZ173178	Spring		1


# Hangers ADM LH/RH manual

1449409X.dwg

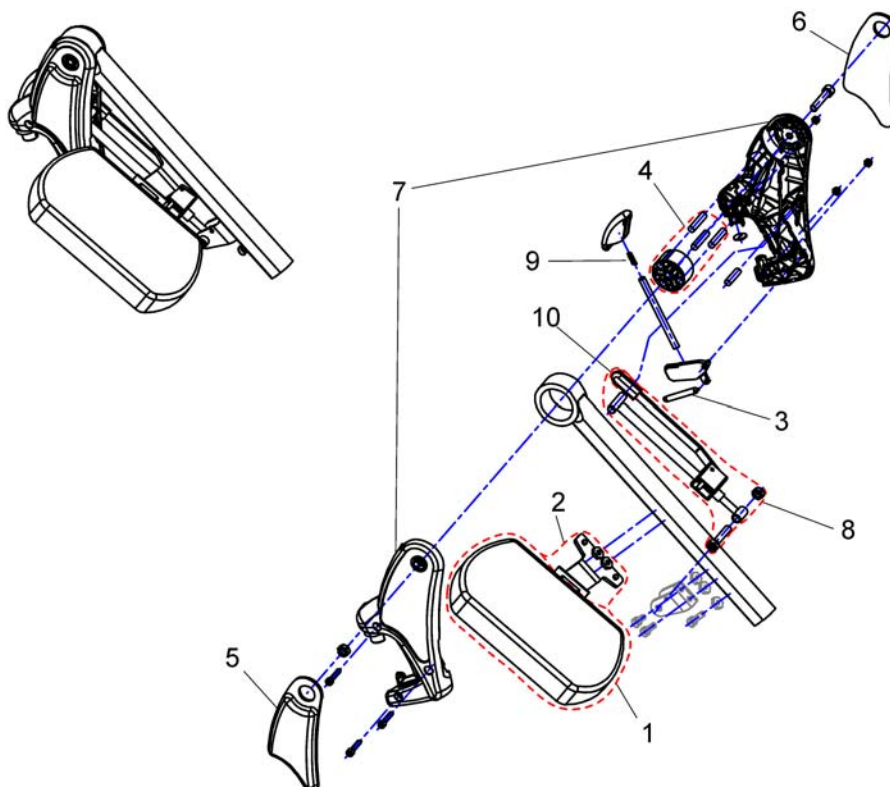

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450410	Knee Upholstery LH		1
6	SP1450448	Knee Upholstery RH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ174130	ADM Adjustment mechanism cpl.		1
9	SPN12196	Spring		1
10	SP1006490	Cap for tripping lever		1
11	SP1449410	Hanger ADM RH cpl.		1
11	SP1449409	Hanger ADM LH cpl.		1


# Hangers ADE/ADP LH/RH electric 07.2005 =>

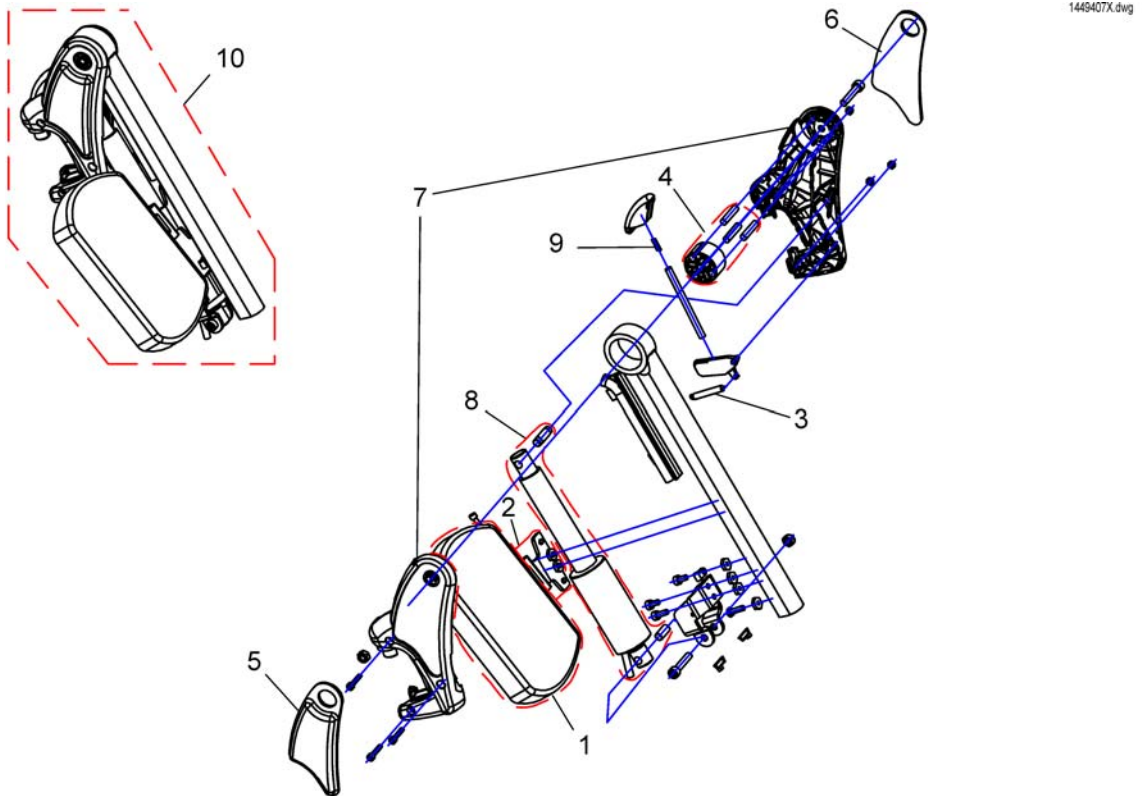

Pos.	Part number	Description	Valid from → until	Qty
1	6301333	Legrest calf pad		1
2	SP1450447	Calf Pad Holder Left Turnable, cpl.		1
2	SP1450446	Calf Pad Holder Right Turnable, cpl.		1
3	SPZ173178	Spring		1
4	SPZ174134	Floating Cylinder With Pin		1
5	SP1450448	Knee Upholstery RH		1
6	SP1450410	Knee Upholstery LH		1
7	SP1449293	Ad-Cover cpl.		1
8	SPZ151006	Actuator with circuit board, preassembled		1
9	SPN12196	Spring		1
10	SP1449408	Hanger ADP RH cpl.		1
10	SP1449407	Hanger ADP LH cpl.		1


# Footrests Vari, Flip-Up - 01.08.10 =>

152964X.jpg

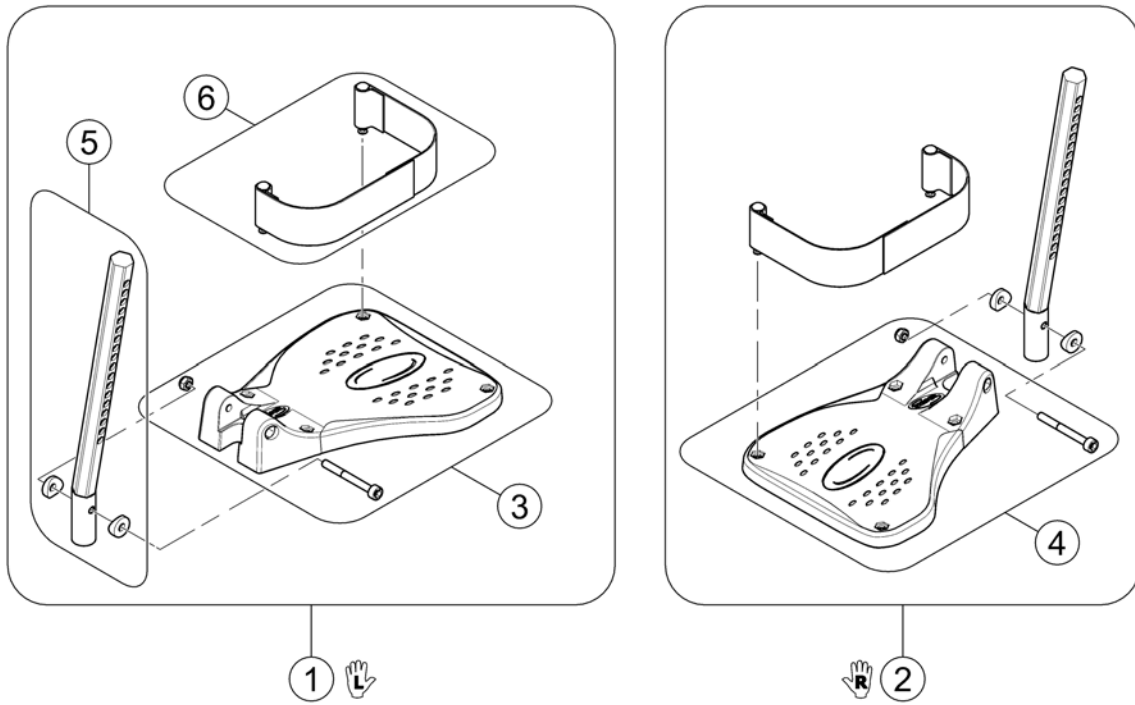

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541575	Footplate flip up, LH, cpl.		1
2	SP1541574	Footplate flip up, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1543045	Plug-in tube LH/RH		1
6	SP1442467	Heel Strap cpl.		1


## Footrests Vari, Flip-Up & angle adjustable - 01.08.10 =>

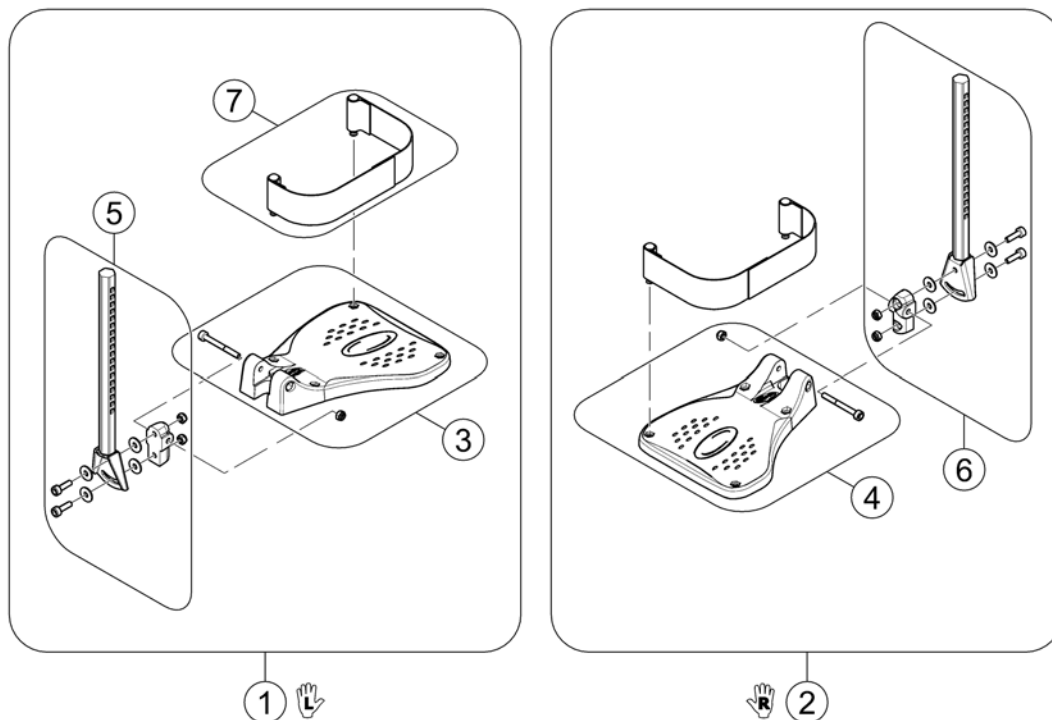

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541580	Footplate flip up, angle adjustable, LH, cpl.		1
2	SP1541579	Footplate flip up, angle adjustable, RH, cpl.		1
3	SP1529645	FOOTPLATE LH		1
4	SP1529644	FOOTPLATE RH		1
5	SP1423204	FOOT PLATE TUBE, ANGLE ADJUSTMENT LH CPL.		1
6	SP1423203	FOOT PLATE TUBE, ANGLE ADJUSTMENT RH CPL.		1
7	SP1442467	Heel Strap cpl.		1


## Footrests Vari, angle & depth adjustable

1583898 iso

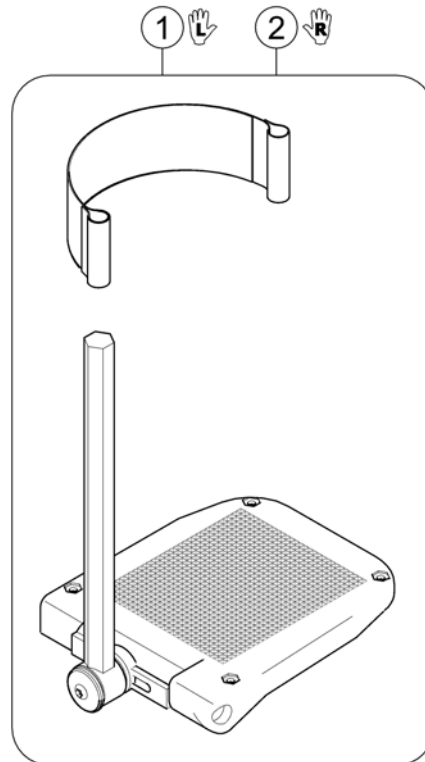

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419344	Footplate LH Anlge And Depth Adjustable, With Support Tube cpl.		1
2	SP1419345	Footplate RH Anlge And Depth Adjustable, With Support Tube cpl.		1




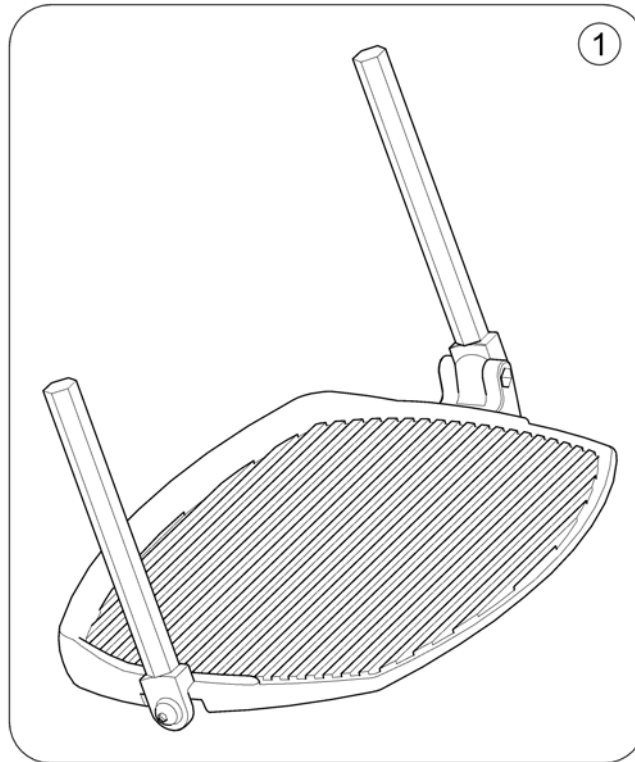

Pos.	Part number	Description	Valid from → until	Qty
1	SP1423156	Footboard, pre-assembled		1

# Support Centr. Legrest LNX man. - Modulite Seat


1563891.jpg

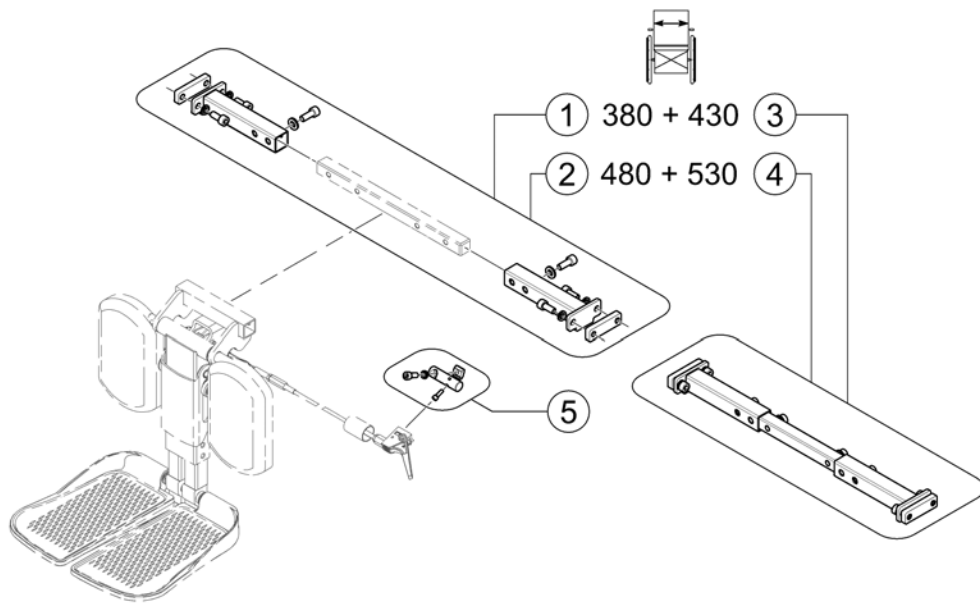

Pos.	Part number	Description	Valid from → until	Qty
1	SP1579989	Support, Centr. Legrest man. LNX - SW 380+430mm		1
2	SP1579990	Support, Centr. Legrest man. LNX - SW 480+530mm		1
3	SP1579991	Support, Centr. Legrest man. LNX - SW 380+430mm, cpl.		1
4	SP1579992	Support, Centr. Legrest man. LNX - SW 480+530mm, cpl.		1
5	SP1579988	Support, Lever, cpl.		1


# Support Centr. Legrest LNX el. - Modulite Seat

1583893.kim

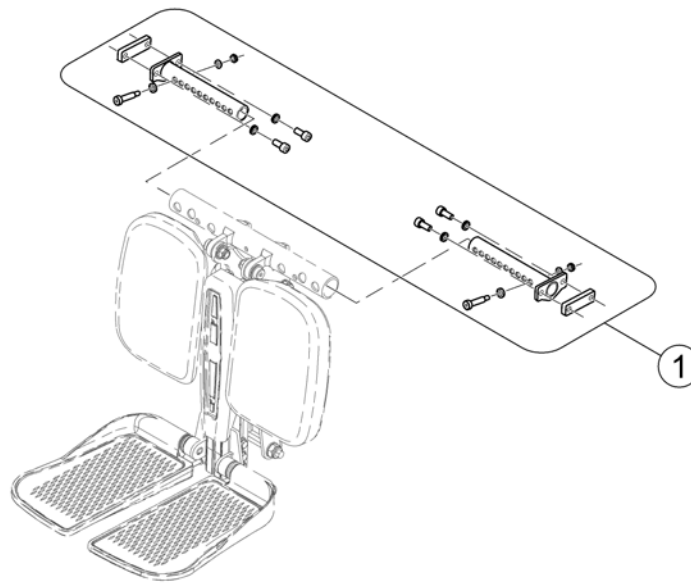

Pos.	Part number	Description	Valid from → until	Qty
1	SP1575144	Support, Centr. Legrest el. LNX - Modulite		1


## Centr. Legrest man.

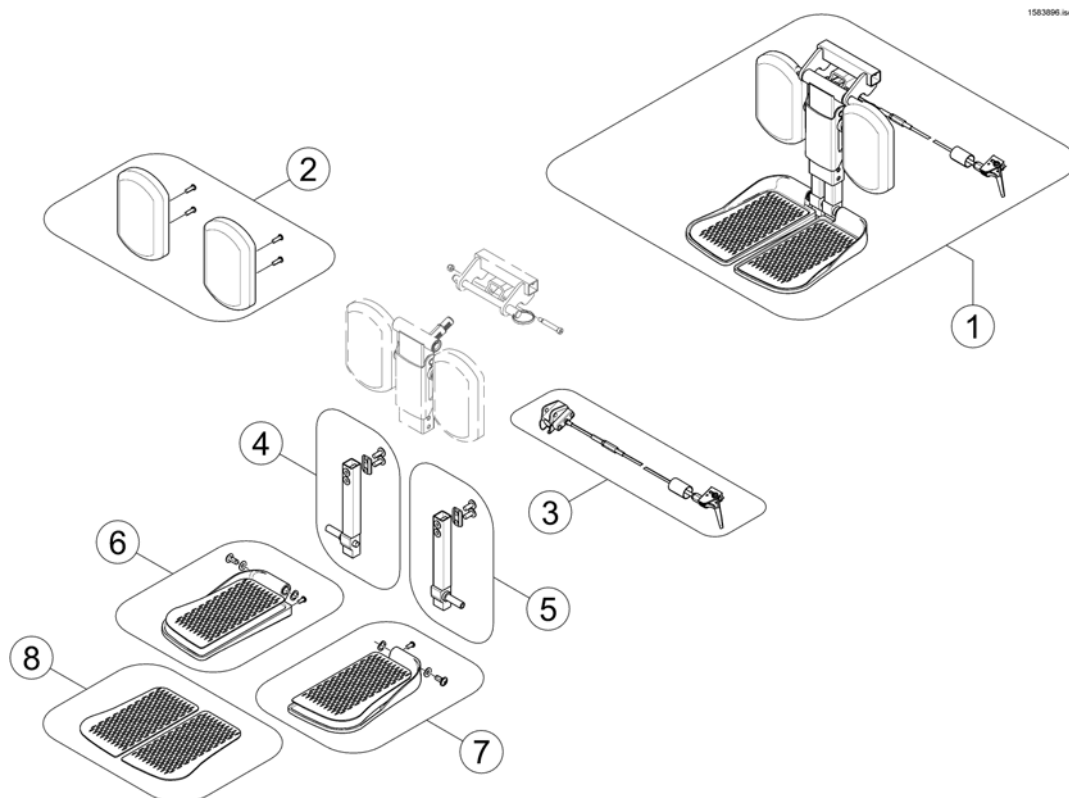

Pos.	Part number	Description	Valid from → until	Qty
1	SP1537926	Central Legrest manual, cpl.		1
2	SP1163373	Calfplates (Pair)		1
3	SP1121079	Bowden cable central legrest, manual		1
4	SP1161986	Kit, extension tube, cpl. RH		1
5	SP1161987	Kit, extension tube, cpl. LH		1
6	SP1161990	Footplate, cpl. RH		1
7	SP1161991	Footplate, cpl. LH		1
8	SP1163375	Footrest pad (Pair)		1

## Centr. Legrest LNX el.

1583894.000

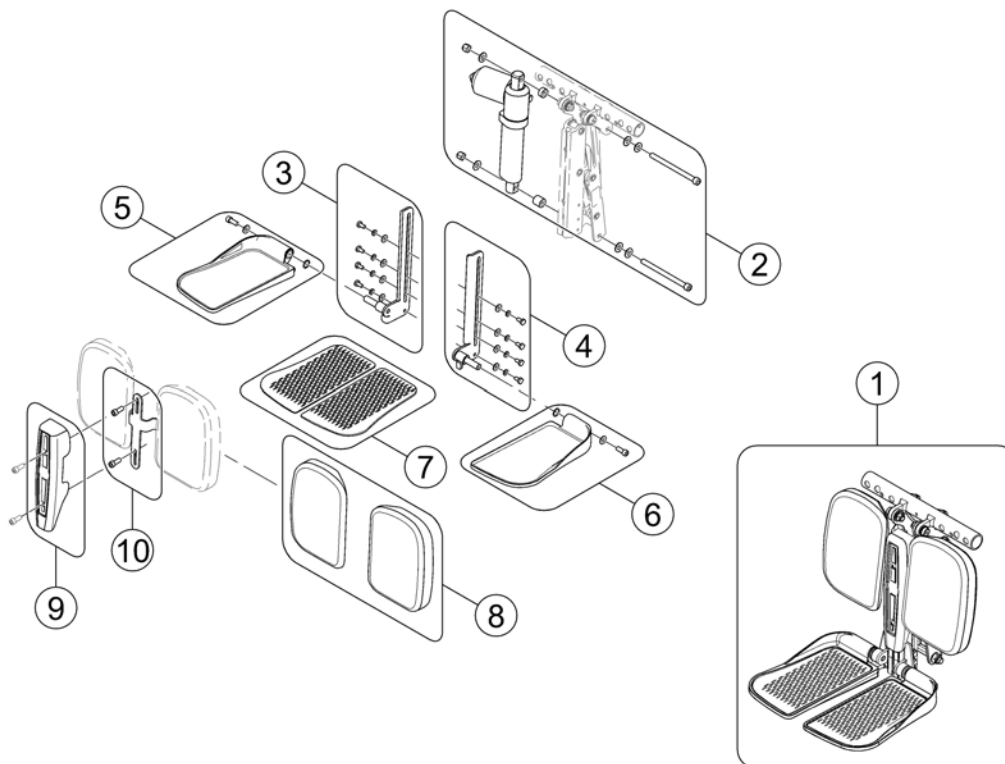


Pos.	Part number	Description	Valid from → until	Qty
-	SP1541677	Adaptercable 300mm <i>For chairs with lifter</i>		1
-	SP1543012	Adaptercable 1200mm <i>For chairs without lifter</i>		1
-	SP1571959	Centr. Legrest el. LNX for Modulite cpl. (incl. Support)		1
1	SP1571882	Centr. Legrest LNX el.		1
2	SP1543031	Actuator, complete		1
3	SP1543035	Footplate mount, RH		1
4	SP1543034	Footplate mount, LH		1
5	SP1543037	Footplate, RH		1
6	SP1543036	Footplate, LH		1
7	SP1163375	Footrest pad (Pair)		1
8	SP1543033	Calfplates (Pair)		1
9	SP1543032	Cover		1
10	SP1579985	Calf Plate Bracket		1

## Max Seat

Pos.	Part number	Description	Valid from → until	Qty
-		Legrest Support 5cm, Vari series		0
-		Hangers Vari A LH/RH 04.2009 =>		0
-		Hangers Vari F LH/RH		0
-		Hangers ADM LH/RH manual		0
-		Hangers ADE/ADP LH/RH electric 07.2005 =>		0
-		Footrests Vari, Flip-Up - 01.08.10 =>		0
-		Footrests Vari, Flip-Up & angle adjustable - 01.08.10 =>		0
-		Footrests Vari, angle & depth adjustable		0
-		Footboard Vari, pre-assembled		0
-		Support Centr. Legrest man. LNX - Max Seat		0
-		Support Centr. Legrest LNX el. - Max Seat		0
-		Centr. Legrest man.		0
-		Centr. Legrest LNX el.		0
-		Fixed footboard		0
-		Calf plates		0

## Support Centr. Legrest man. LNX - Max Seat


1583889 (en)

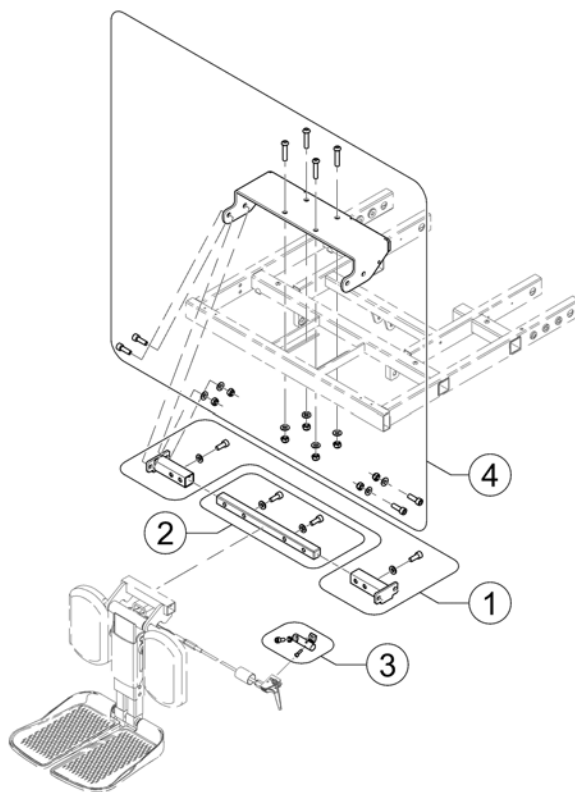

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583887	Mounting Kit, Central Legrest LNX man. - Max Seat		1
2	SP1576712	Insert, Centr. Legrest LNX		1
3	SP1579988	Support, Lever, cpl.		1
4	SP1580689	Upper Support, Centr. Legrest LNX - Max Seat		1

## Support Centr. Legrest LNX el. - Max Seat


1583890.iss

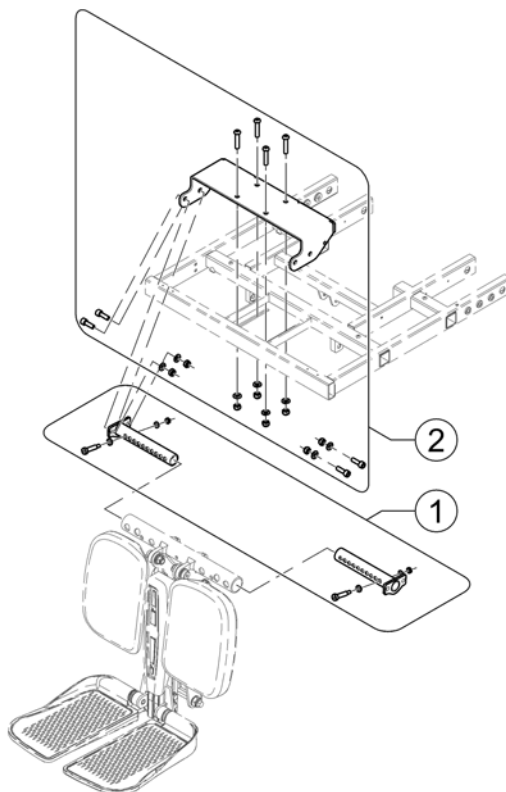

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583888	Mounting Kit, Central Legrest LNX el. - Max Seat		1
2	SP1580689	Upper Support, Centr. Legrest LNX - Max Seat		1




## Fixed footboard

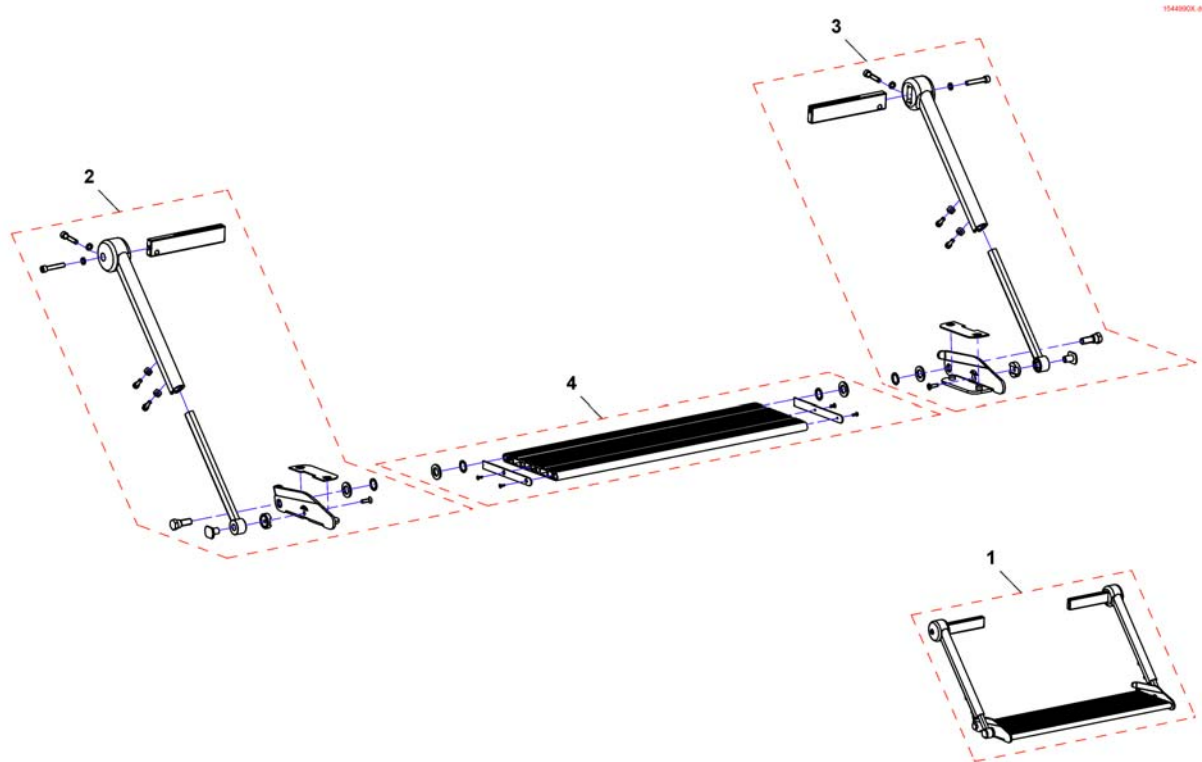

Pos.	Part number	Description	Valid from → until	Qty
1	SP1544990	Fixed footboard SW61		1
1	SP1545454	Fixed footboard SW55		1
1	SP1545455	Fixed footboard SW49		1
2	SP1552445	Footboard bracket RH		1
3	SP1552446	Footboard bracket LH		1
4	SP1552466	Footboard SW49		1
4	SP1552465	Footboard SW55		1
4	SP1552447	Footboard SW61		1

# Calf plates

1541635X.iso

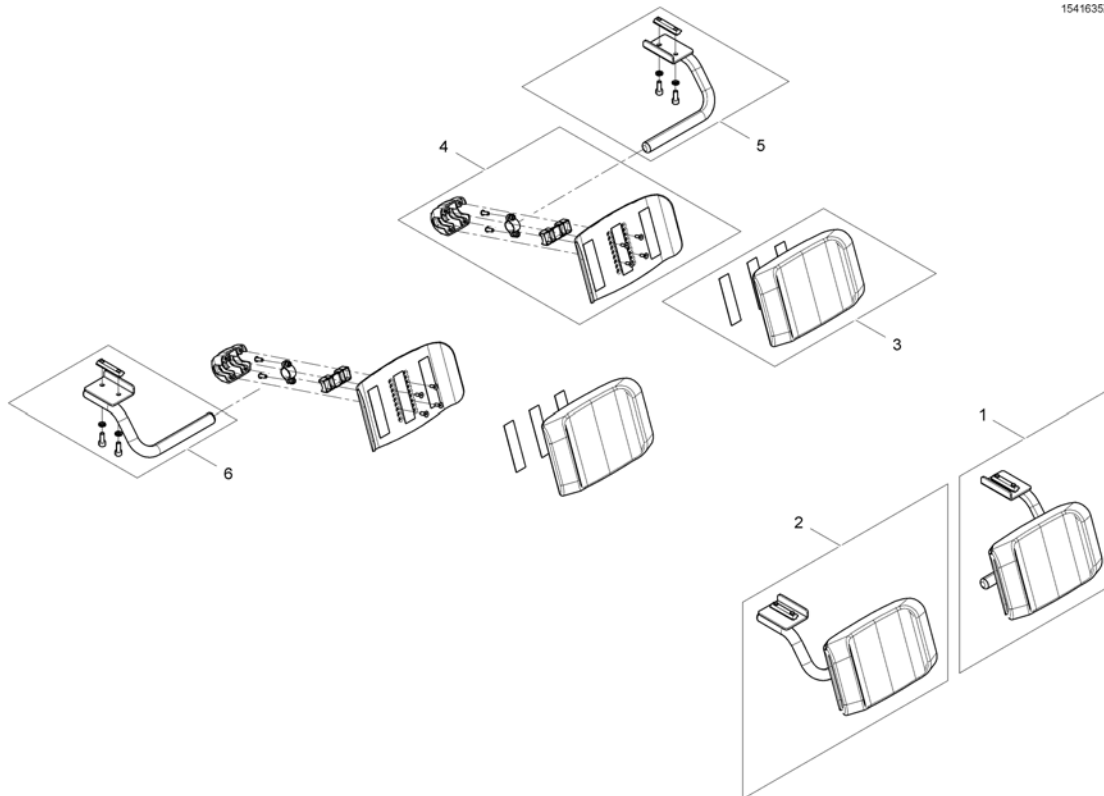



Pos.	Part number	Description	Valid from → until	Qty
1	SP1544875	Calf pad, SW 490mm, LH, cpl.		1
1	SP1541635	Calf pad, SW 550mm & 610mm, LH, cpl.		1
2	SP1544876	Calf pad, SW 490mm, LH, cpl.		1
2	SP1541636	Calf pad, SW 550mm & 610mm, RH, cpl.		1
3	SP1541672	Calf Pad SW 490mm		1
3	SP1541491	Calf Pad SW 550mm & 610mm		1
4	SP1541669	Calf pad bracket SW 490mm, LH/RH, cpl.		1
4	SP1541489	Calf pad bracket SW 550mm & 610mm, LH/RH, cpl.		1
5	SP1541633	Calf pad bracket, LH, cpl.		1
6	SP1541634	Calf pad bracket, RH, cpl.		1

**Legrests - Narrow Base**

Pos.	Part number	Description	Valid from → until	Qty
-		Legrest Support 5cm, Vari series		0
-		Hangers Vari A LH/RH 04.2009 =>		0
-		Hangers Vari F LH/RH		0
-		Hangers ADM LH/RH manual		0
-		Hangers ADE/ADP LH/RH electric 07.2005 =>		0
-		Footrests Vari, Flip-Up - 01.08.10 =>		0
-		Footrests Vari, Flip-Up & angle adjustable - 01.08.10 =>		0
-		Footrests Vari, angle & depth adjustable		0
-		Support Centr. Legrest LNX man. - Modulite Seat		0
-		Support Centr. Legrest LNX el. - Modulite Seat		0
-		Centr. Legrest man.		0
-		Centr. Legrest LNX el.		0

# Armrests


SPCTDX\_SP220.iso

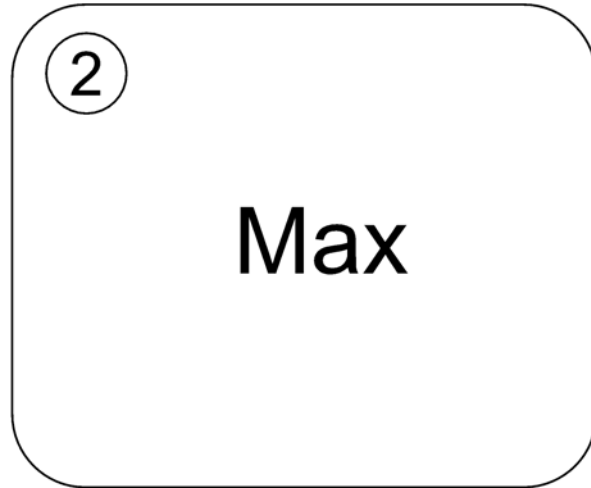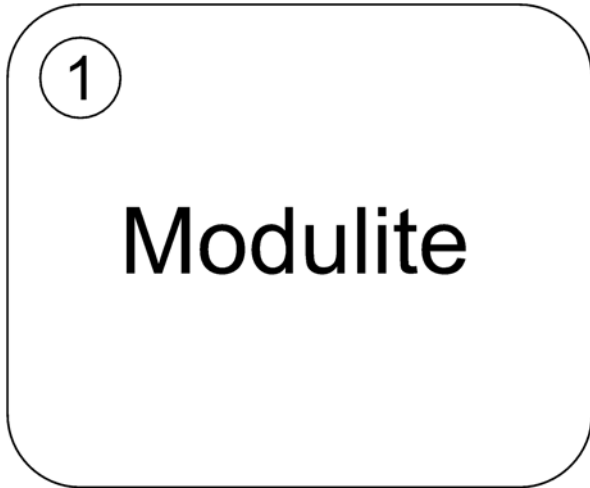

Pos.	Part number	Description	Valid from → until	Qty
-		Armrests for Modulite Seat		0
-		Armrests for Max Seat		0

## Armrests for Modulite Seat


1571738.000

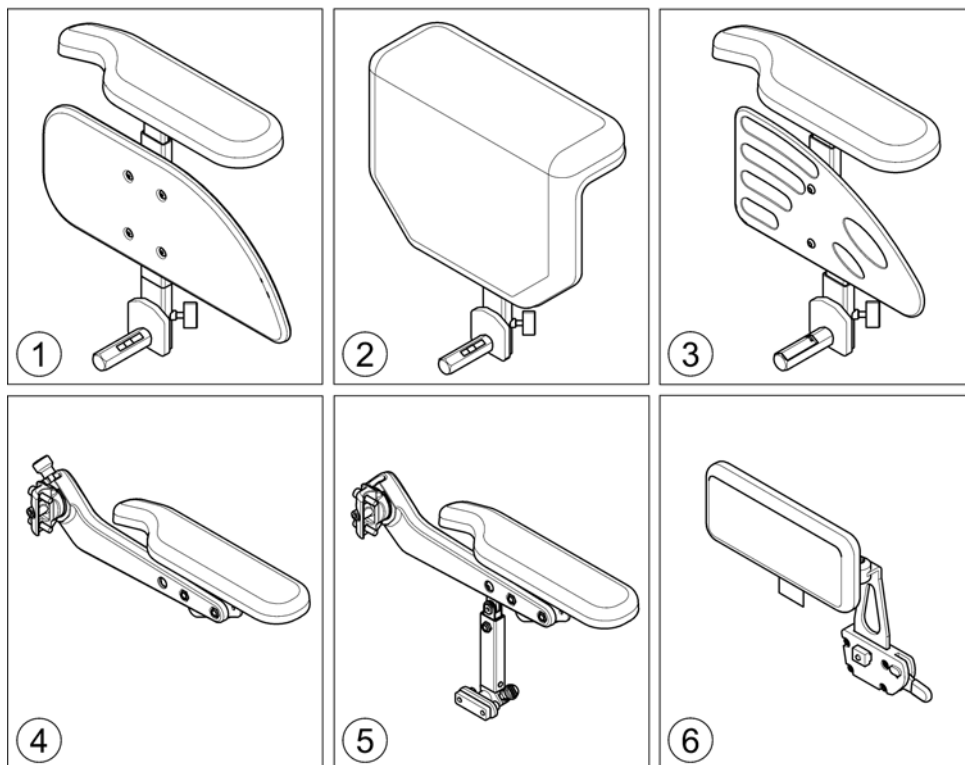

Pos.	Part number	Description	Valid from → until	Qty
1	-	T-Armrest [2191+2195/2196/2197]		1
2	-	Padded Armrest [2198]		1
3	-	Inrigged Armrest [2183+2195/2196/2197/2199]		0
4	-	Flip-Up Armrest [2193+2195/2196/2197/2199]		0
5	-	Following Armrest [2192/2194+2195/2196/2197/2199]		0
6	-	Hip Support [2184/2185/2186/2187/2188/2189]		0

# T-Armrest


1572570.iso

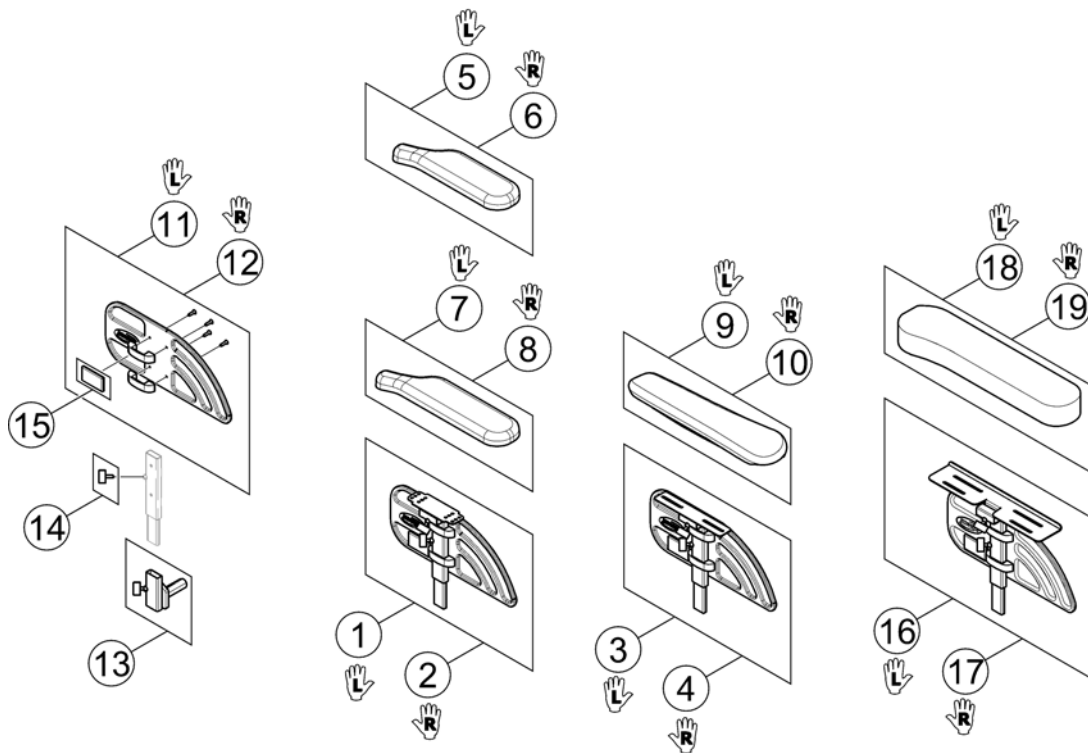

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570318	Siderest LH (for Standard & Extended Armpad)		1
2	SP1570368	Siderest RH (for Standard & Extended Armpad)		1
3	SP1578114	Siderest LH (for Ergonomic Armpad)		1
4	SP1578115	Siderest RH (for Ergonomic Armpad)		1
5	SP1443004	Armpad, Standard (325mm), LH		1
6	SP1449043	Armpad, Standard (325mm), RH		1
7	SP1528846	Armpad, Extended (375mm), LH		1
8	SP1528845	Armpad, Extended (375mm), RH		1
9	SP1571589	Armpad, Ergonomic, LH		1
10	SP1571588	Armpad, Ergonomic, RH		1
11	SP1572940	Skirt Guard LH		1
12	SP1572948	Skirt Guard RH		1
13	SP1567749	Siderest Bracket - LH/RH		1
14	SPL10024	Wing Screw M6x16		1
15	SPP31298	Reflector, yellow		1
16	SP1585205	Siderest LH (for Soft Armpad)		1
17	SP1585206	Siderest RH (for Soft Armpad)		1
18	SP1585068	Armpad, Soft, LH		1
19	SP1585069	Armpad, Soft, RH		1

## Padded Armrest


1573214.00

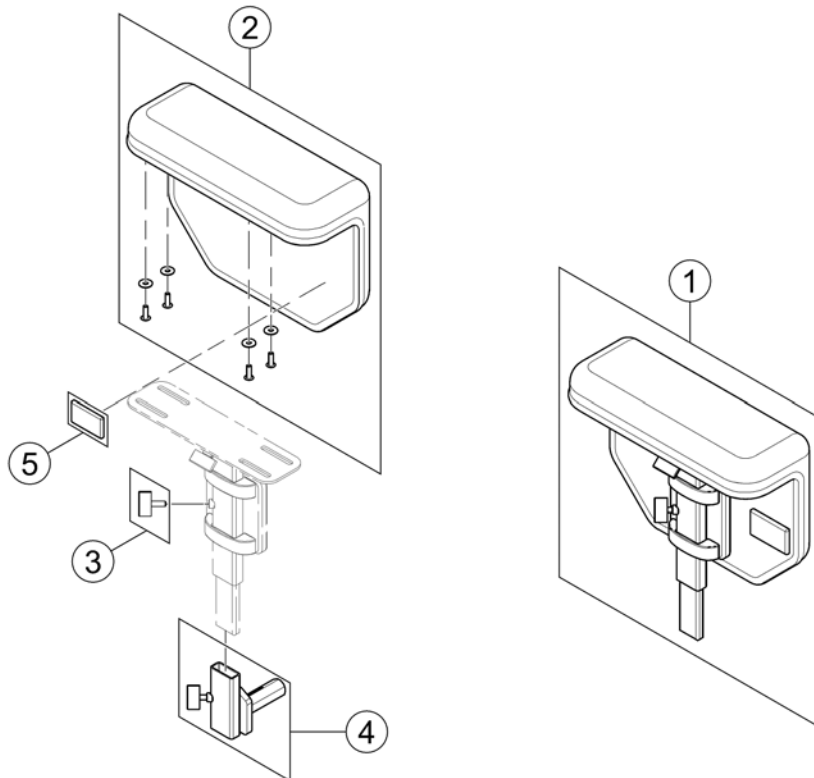

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571987	Padded Armrest RH/LH, cpl.		1
2	SP1571962	Armpad, padded - RH/LH with integrated cloth guard		1
3	SPL10024	Wing Screw M6x16		1
4	SP1567749	Siderest Bracket - LH/RH		1
5	SPP31298	Reflector, yellow		1

# Inrigged Armrest


1585180 iso

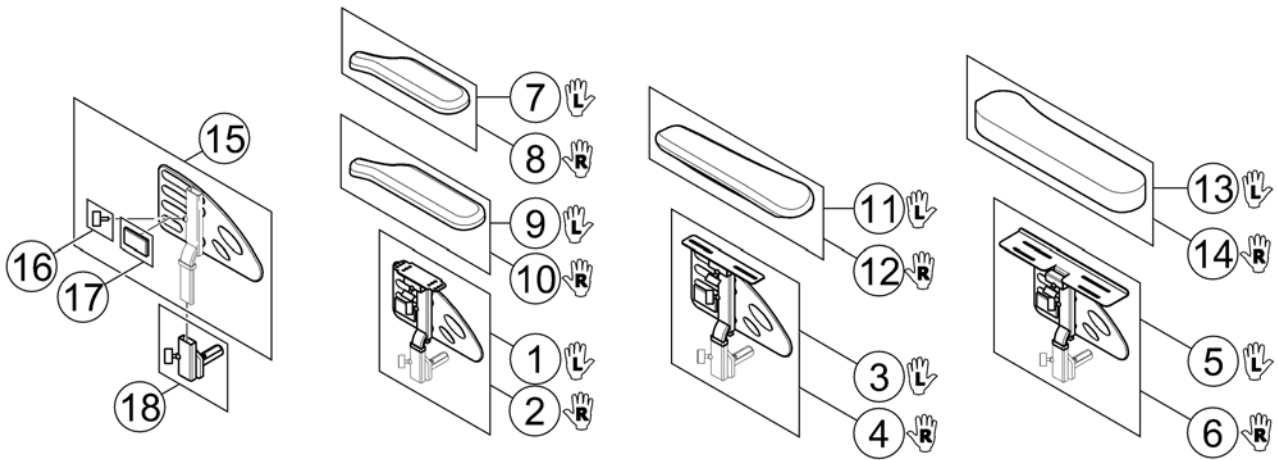

Pos.	Part number	Description	Valid from → until	Qty
1	SP1585448	Armrest LH (for Standard & Extended Armpad)		1
2	SP1585449	Armrest RH (for Standard & Extended Armpad)		1
3	SP1585450	Armrest LH (for Ergonomic Armpad)		1
4	SP1585451	Armrest RH (for Ergonomic Armpad)		1
5	SP1585452	Armrest LH (for Soft Armpad)		1
6	SP1585453	Armrest RH (for Soft Armpad)		1
7	SP1443004	Armpad, Standard (325mm), LH		1
8	SP1449043	Armpad, Standard (325mm), RH		1
9	SP1528846	Armpad, Extended (375mm), LH		1
10	SP1528845	Armpad, Extended (375mm), RH		1
11	SP1571589	Armpad, Ergonomic, LH		1
12	SP1571588	Armpad, Ergonomic, RH		1
13	SP1585068	Armpad, Soft, LH		1
14	SP1585069	Armpad, Soft, RH		1
15	SP1578119	Skirt Guard LH/RH		1
16	SPL10024	Wing Screw M6x16		1
17	SPP31298	Reflector, yellow		1
18	SP1567749	Siderest Bracket - LH/RH		1




## Flip-Up Armrest

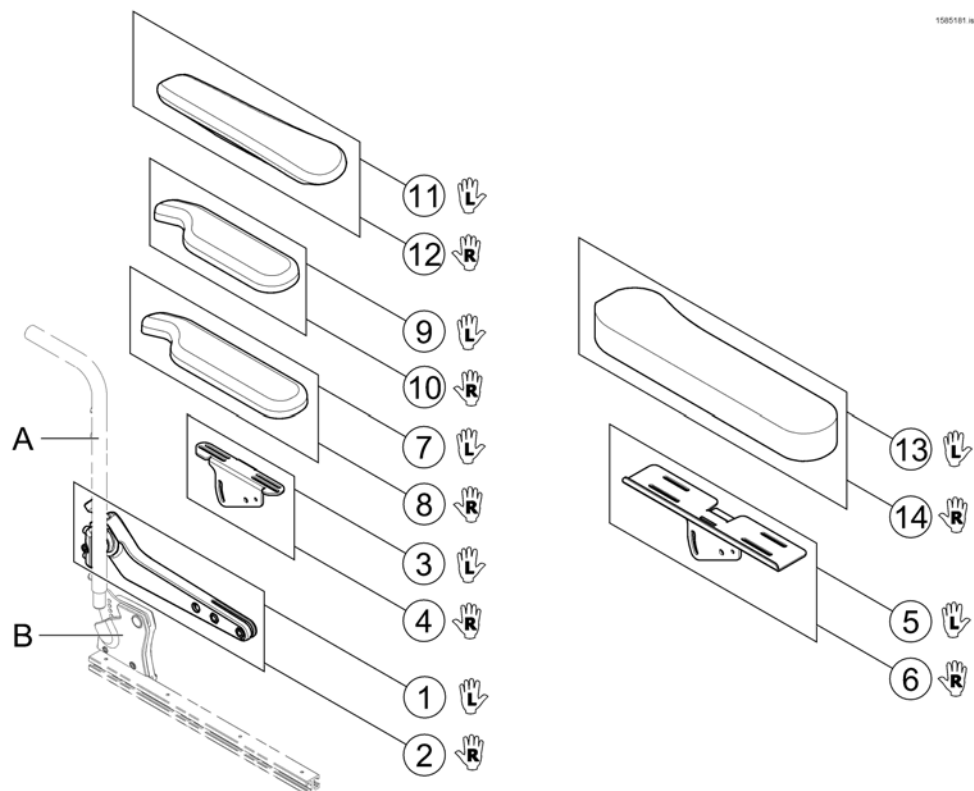

1580181 Rev

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1582668	Flip-Up Armrest, LH	2015-01-19 →	1
2	SP1582644	Flip-Up Armrest, RH	2015-01-19 →	1
3	SP1585412	Armpad Support (Standard, Extended, Ergonomic), LH		1
4	SP1585413	Armpad Support (Standard, Extended, Ergonomic), RH		1
5	SP1585414	Armpad Support (Soft), LH		1
6	SP1585415	Armpad Support (Soft), RH		1
7	SP1528846	Armpad, Extended (375mm), LH		1
8	SP1528845	Armpad, Extended (375mm), RH		1
9	SP1443004	Armpad, Standard (325mm), LH		1
10	SP1449043	Armpad, Standard (325mm), RH		1
11	SP1571589	Armpad, Ergonomic, LH		1
12	SP1571588	Armpad, Ergonomic, RH		1
13	SP1585068	Armpad, Soft, LH		1
14	SP1585069	Armpad, Soft, RH		1

## Following Armrest

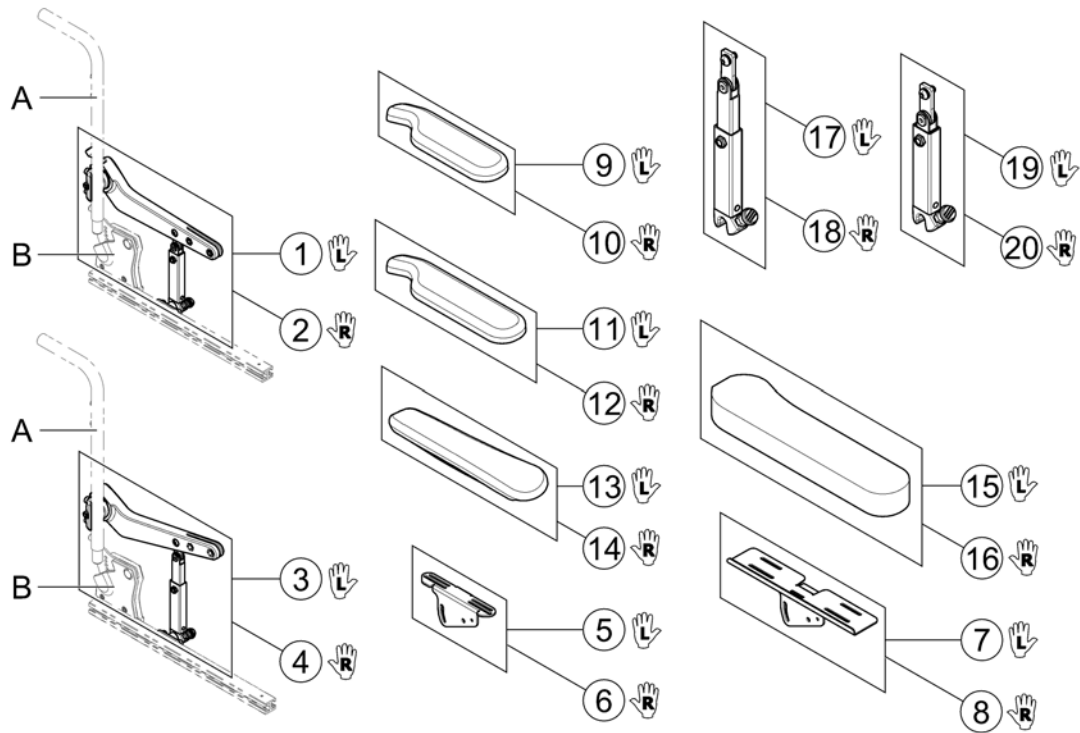


1585182 No

Pos.	Part number	Description	Valid from → until	Qty
-	-	Retrofitting the armrests on chairs built before 15-03-27 also requires new Pos. A & B and Sling Back with respective cut-outs! !!!		0
1	SP1585263	Armrest LH cpl. - Short Support	2015-01-19 →	1
2	SP1585264	Armrest RH cpl. - Short Support	2015-01-19 →	1
3	SP1585267	Armrest LH cpl. - Long Support	2015-01-19 →	1
4	SP1585268	Armrest RH cpl. - Long Support	2015-01-19 →	1
5	SP1585412	Ampad Support (Standard, Extended, Ergonomic), LH		1
6	SP1585413	Ampad Support (Standard, Extended, Ergonomic), RH		1
7	SP1585414	Ampad Support (Soft), LH		1
8	SP1585415	Ampad Support (Soft), RH		1
9	SP1443004	Ampad, Standard (325mm), LH		1
10	SP1449043	Ampad, Standard (325mm), RH		1
11	SP1528846	Ampad, Extended (375mm), LH		1
12	SP1528845	Ampad, Extended (375mm), RH		1
13	SP1571589	Ampad, Ergonomic, LH		1
14	SP1571588	Ampad, Ergonomic, RH		1
15	SP1585068	Ampad, Soft, LH		1
16	SP1585069	Ampad, Soft, RH		1
17	SP1583156	Amrest Support, long, LH		1
18	SP1583155	Amrest Support, long, RH		1
19	SP1583154	Amrest Support, short, LH		1
20	SP1582639	Amrest Support, short, RH		1

# Hip Support


1585183 Rev

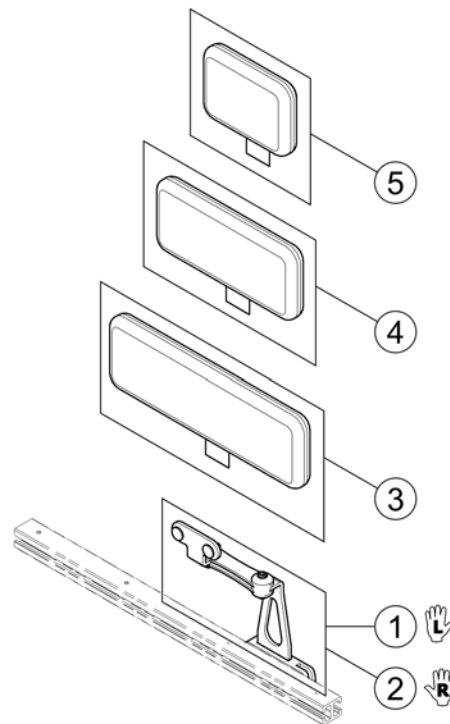

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584411	Hip Support, Quick Release bracket, LH		1
2	SP1584410	Hip Support, Quick Release bracket, RH		1
3	SP1584414	Hip Support Pad, large (330x100 mm)		1
4	SP1584413	Hip Support Pad, medium (230x100 mm)		1
5	SP1584412	Hip Support Pad, small (140x100 mm)		1

## Armrests for Max Seat


1583900.kim

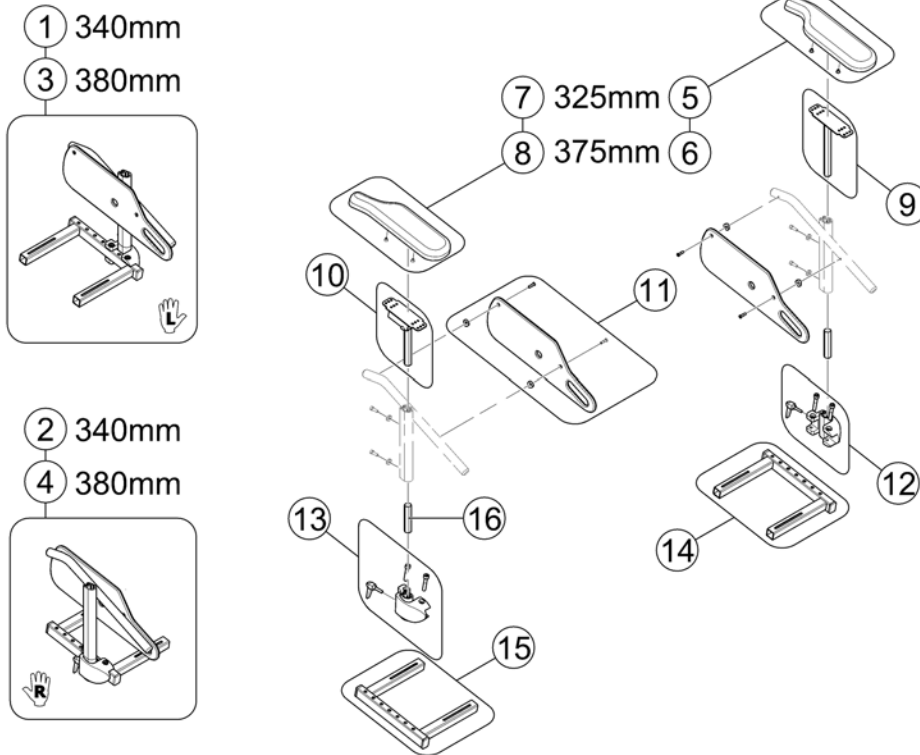

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583906	Siderest 340mm LH, without Armrest - Max Seat		1
2	SP1583907	Siderest 340mm RH, without Armrest - Max Seat		1
3	SP1583908	Siderest 380mm LH, without Armrest - Max Seat		1
4	SP1583909	Siderest 380mm RH, without Armrest - Max Seat		1
5	SP1443004	Armpad, Standard (325mm), LH		1
6	SP1528846	Armpad, Extended (375mm), LH		1
7	SP1449043	Armpad, Standard (325mm), RH		1
8	SP1528845	Armpad, Extended (375mm), RH		1
9	SP1510857	Armpad holder LH		1
10	SP1510856	Armpad holder RH		1
11	SP1428078	Skirtguard, seat depth 410mm		1
12	SP1541647	Clamp, LH, cpl.		1
13	SP1541646	Clamp, RH, cpl.		1
14	SP1537209	Bracket, cpl., LH		1
15	SP1537208	Bracket, cpl., RH		1
16	1422092-0002	CONNECTION PIECE		1


# Castors

1583268.kso

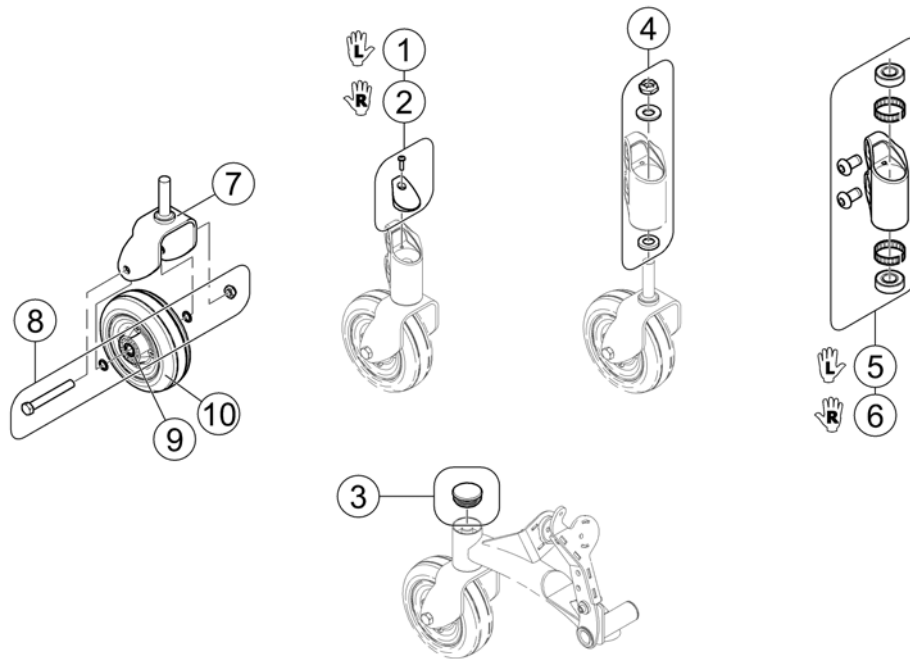

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523141	Kit, Cap with Screw, Front Head Tube -Left		1
2	SP1523140	Kit, Cap with Screw, Front Head Tube -Right		1
3	SP1110709	Plug Button, Head Tube Black		1
4	SP1522999	Kit, Fork Mounting Hardware		1
5	SP1523139	Kit, Front Head Tube Assembly-Left		1
6	SP1523094	Kit, Front Head Tube Assembly-Right		1
7	SP1523016	Fork and Stem For TRRO/TRBKTS Option		1
8	SP1537431	Kit, Castor Mounting Hardware		1
9	SP1522935	Spacer, Bearing (7/16 x 5/8 x 1-3/4")		1
10	SP1175881	6x2" Wheel with bearings, silver hub		1


## Drive Wheels 14"

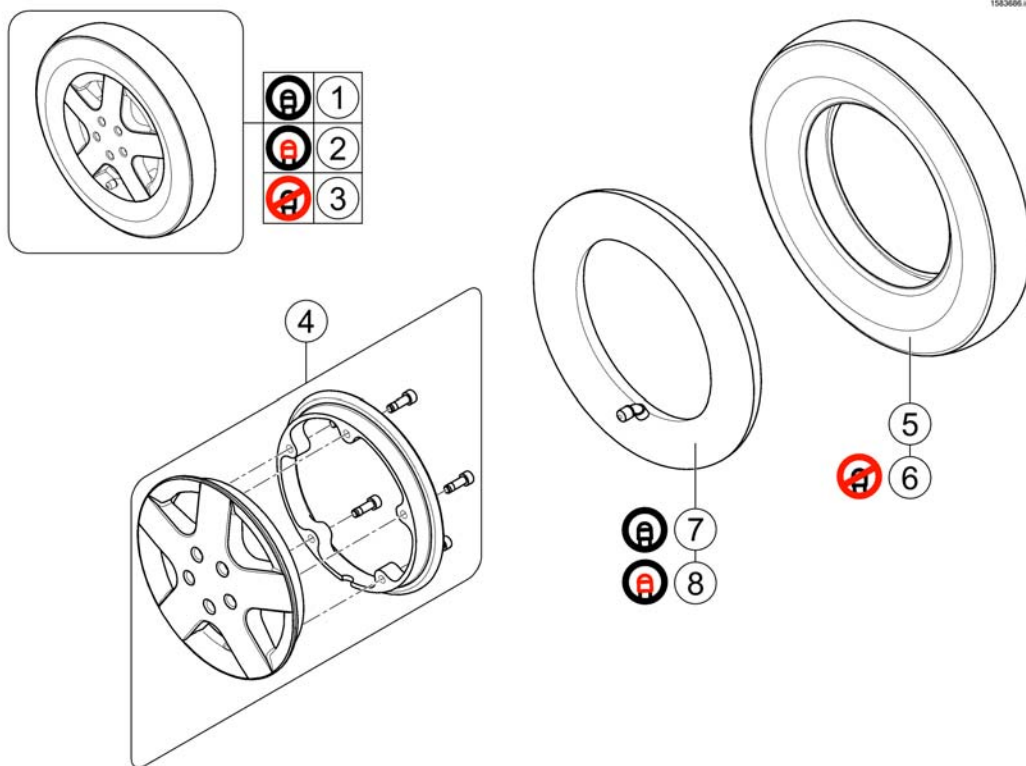

Pos.	Part number	Description	Valid from → until	Qty
1	SP1532414	Wheel 14 x 3.00" pneumatic, silver, complete		1
2	SP1532415	Wheel 14 x 3.00" puncture protected, silver, complete		1
3	SP1532416	Wheel 14 x 3.00" puncture proof, silver, complete		1
4	SP1541373	Rim Kit, complete		1
5	SP1522940	Tire 14 x 3.00"		1
6	SP1141342	Tire 14 x 3.00" puncture proof		1
7	SP1039651	Tube 14"		1
8	SP1541374	Tube 14" puncture protected		1


**Wheel Hub 5-hole**

1583687 Rev

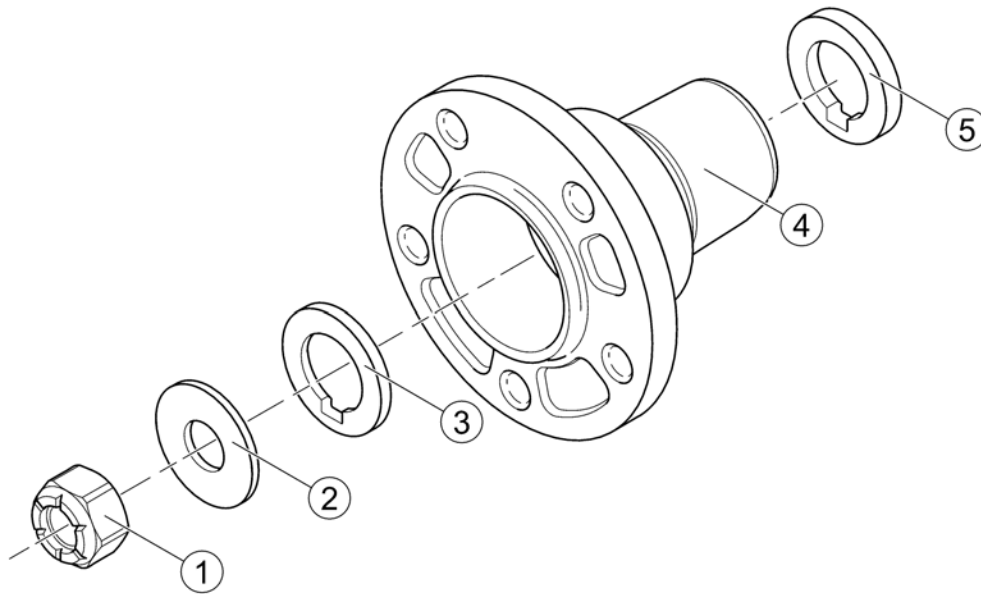

Pos.	Part number	Description	Valid from → until	Qty
1	F00108	Hexagon nut DIN985-M12x1,5 zc		1
2	G00104	Washer DIN125-Ø13 zc		1
3	SP1544342	Washer with feather nut, 5mm thick, d=19mm		1
4	SP1163728	Wheel hub 5-hole		1
5	SP1163729	Spacer, Hub, 5mm thick		1

# Propulsion SSD - Details

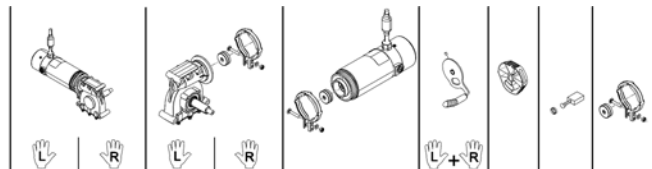

			L	R	L	R		L+R			
Bora/ SpectraXTR	6km/h	c ACS/ Shark	1	2	33	34	37	---	51	45	
		VR2	3	4	33	34	43	---	51	45	
		R-Net	5	6	33	34	41	---	51	45	
	10km/h	d ACS/ Shark	7	8	31	32	36	---	51	45	
		VR2	9	10	31	32	40	---	51	45	
		R-Net	11	12	31	32	42	---	51	45	
BoraHD/ Spectra XTR2HD	6km/h	ACS/ Shark	54	55	33	34	39	---	51	46	
	10km/h		56	57	31	32	38	---	51	46	
	6km/h	R-Net	58	59	33	34	62	---	51	46	
	10km/h		60	61	31	32	63	---	51	46	
FDX	6km/h	ACS/ Shark	13	14	33	34	37	53	---	45	
	10km/h		15	16	31	32	38	53	---	46	
Kite	6km/h	ACS/ Shark	17	18	33	34	37	---	51	45	
	10km/h		19	20	31	32	36	---	51	45	
Kite HD	6km/h	ACS/ Shark	21	22	33	34	39	---	51	46	
	10km/h		23	24	31	32	38	---	51	46	
Storm4/ Storm4 X-plore	6km/h	ACS/ Shark	47	48	34	33	39	---	51	46	
	10km/h		27	28	32	31	38	---	51	46	
TDX-SP	6km/h	ACS/ Shark	a	25	26	34	33	39	53	---	46
	8km/h		b	29	30	34	33	38	53	---	46
	10km/h		c	49	50	32	31	38	53	---	46
TDX-SP Narrow	6km/h	ACS/ Shark	25	26	34	33	39	53	---	46	
	10km/h		d	49	50	32	31	38	53	---	46

a) used from 26.09.11 / b) used from 07.09.11 / c) used from 18.04.11 / d) used from 01.08.11

Pos.	Part number	Description	Valid from → until	Qty
-	N23004	Feather key DIN6885-A-6x6x40		1
1	SP1524195	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
2	SP1524194	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
3	SP1541408	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
4	SP1541410	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
5	SP1541409	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
6	SP1541411	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
7	SP1524197	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
8	SP1524196	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
9	SP1541404	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
10	SP1541406	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
11	SP1541405	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
12	SP1541407	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
13	SP1168983	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
14	SP1168984	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
15	SP1168989	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
16	SP1168988	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
17	SP1568147	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1




**Propulsion SSD - Details**

Pos.	Part number	Description	Valid from → until	Qty
18	SP1568148	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 2-pole, 250W, slow, 600mm cable		1
19	SP1568149	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
20	SP1568150	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 2-pole, 250W, fast, 600mm cable		1
21	SP1568151	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
22	SP1568152	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
23	SP1568153	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
24	SP1568208	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
25	SP1536022	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
26	SP1536021	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
27	SP1541366	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =&gt;</i>		1
28	SP1541365	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable <i>S/N 061122610151 =&gt;</i>		1
29	SP1170307	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
30	SP1170306	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
31	SP1163413	Gearbox, LH, i = 18:1		1
32	SP1163412	Gearbox, RH, i = 18:1		1
33	SP1163415	Gearbox, LH, i = 23:1		1
34	SP1163414	Gearbox, RH, i = 23:1		1
35	SP1542953	Clamp band with clutch		1
36	SP1163099	Motor, 2-pole, 250W, fast, 600mm cable		1
37	SP1164046	Motor, 2-pole, 250W, slow, 600mm cable		1
38	SP1165863	Motor, 4-pole, 340W, fast, 760mm cable		1
39	SP1165865	Motor, 4-pole, 340W, slow, 760mm cable		1
40	SP1544840	Motor, 2-pole, 250W, fast, 600mm cable, VR2		1
41	SP1544796	Motor, 2-pole, 250W, slow, 600mm cable, R-Net		1
42	SP1544841	Motor, 2-pole, 250W, fast, 600mm cable, R-Net		1
43	SP1544795	Motor, 2-pole, 250W, slow, 600mm cable, VR2		1
45	SP1540358	Kit, brushes with caps, 2-pole (Set of 2)		1
46	SP1522967	Kit, brushes with caps, 4-pole (Set of 4)		1
47	SP1583821	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable <i>S/N 061122812868 =&gt;</i>		1
48	SP1583822	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable <i>S/N 061122812868 =&gt;</i>		1
49	SP1534060	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
50	SP1534059	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
51	SP1579595	Declutching knob SSD RH/LH (1 pc) <i>Single part. Consider to buy another one for the other side if required. - Replaces SP1168668, SP1168669 &amp; SP1169005</i>		1
53	SP1169004	Declutching lever, complete set for both sides		1
54	SP1552782	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
55	SP1552783	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1

## Propulsion SSD - Details


Pos.	Part number	Description	Valid from → until	Qty
56	SP1552872	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
57	SP1552873	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
58	SP1552832	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable, R-Net		1
59	SP1552833	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable, R-Net		1
60	SP1552836	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable, R-Net		1
61	SP1552837	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable, R-Net		1
62	SP1562498	Motor, 4-Pol, 340W, slow, 760mm cabel, R-Net		1
63	SP1562499	Motor, 4-Pol, 340W, fast, 760mm cabel, R-Net		1

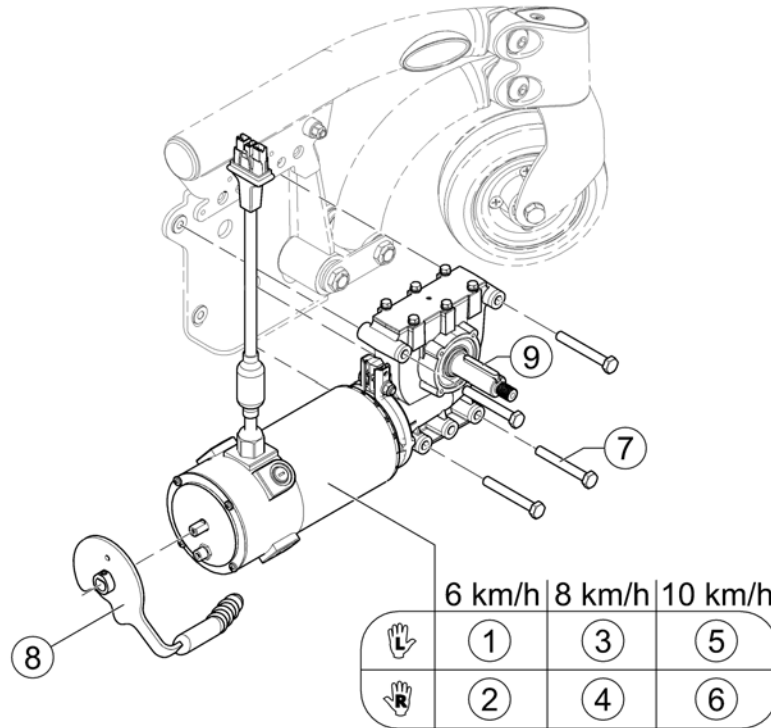



Pos.	Part number	Description	Valid from → until	Qty
1	SP1536022	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
2	SP1536021	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
3	SP1170307	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
4	SP1170306	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, fast, 760mm cable		1
5	SP1534060	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
6	SP1534059	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
7	D00100	Hexagon bolt DIN931-M08x60-8.8 bk		4
8	SP1169004	Declutching lever, complete set for both sides		1
9	N23004	Feather key DIN6885-A-6x6x40		1

# Propulsion SSD - Narrow Base

1563700-00

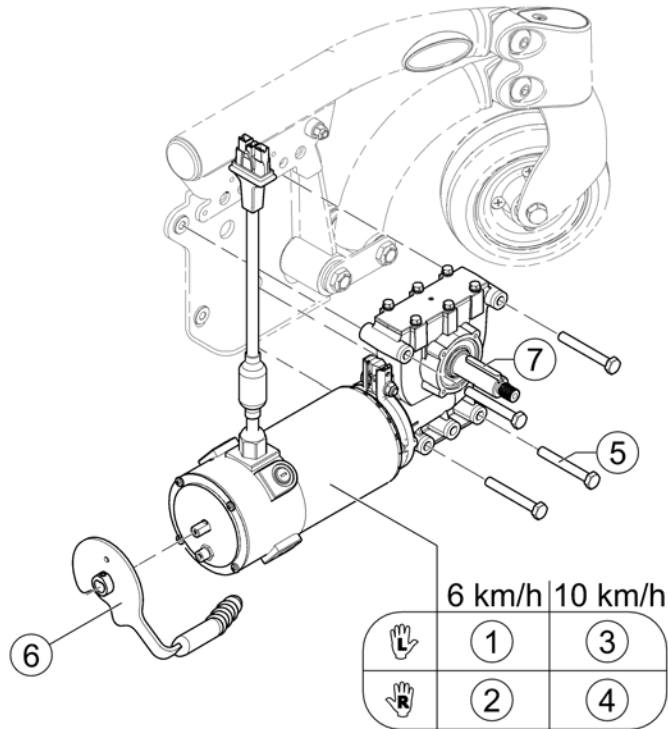


Pos.	Part number	Description	Valid from → until	Qty
1	SP1536022	Propulsion cpl. LH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
2	SP1536021	Propulsion cpl. RH, Gearbox i = 23:1, Motor, 4-pole, 340W, slow, 760mm cable		1
3	SP1534060	Propulsion cpl. LH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
4	SP1534059	Propulsion cpl. RH, Gearbox i = 18:1, Motor, 4-pole, 340W, fast, 760mm cable		1
5	D00100	Hexagon bolt DIN931-M08x60-8.8 bk		4
6	SP1169004	Declutching lever, complete set for both sides		1
7	N23004	Feather key DIN6885-A-6x6x40		1

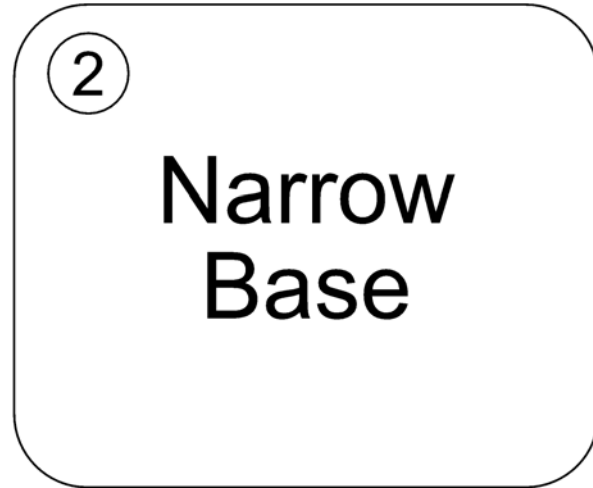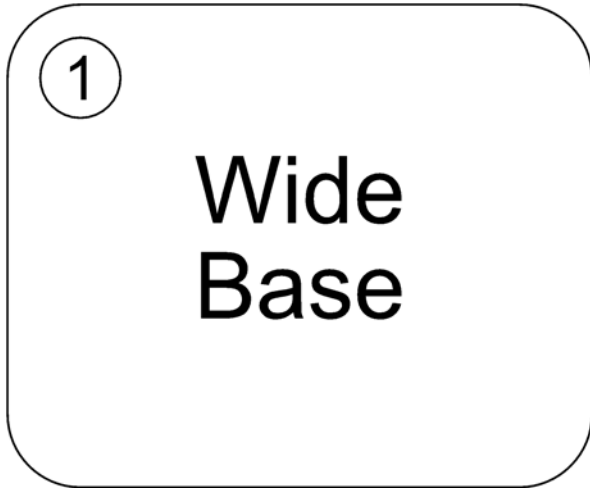

Pos.	Part number	Description	Valid from → until	Qty
1	-	Electronic for Wide Base		1
2	-	Electronic for Narrow Base		1

## Electronic for Wide Base

1500784.sp

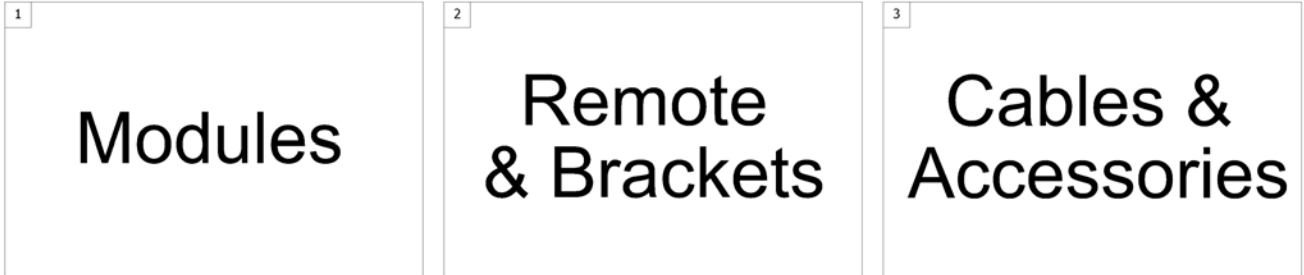


Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic Modules		1
2	-	ACS Remote & Brackets		1
3	-	ACS Cables & Accessories		1

**ACS Electronic Modules**

Pos.	Part number	Description	Valid from → until	Qty
-		Power Modul - ACS		1
-		Bracket for Power Modul Wide Base		1
-		ACT for ACS 2		1
-		Bracket for ACT Wide Base		1
-		Light Board		1
-		G-Trac-sensor		1

## Power Modul - ACS

1583335.iso

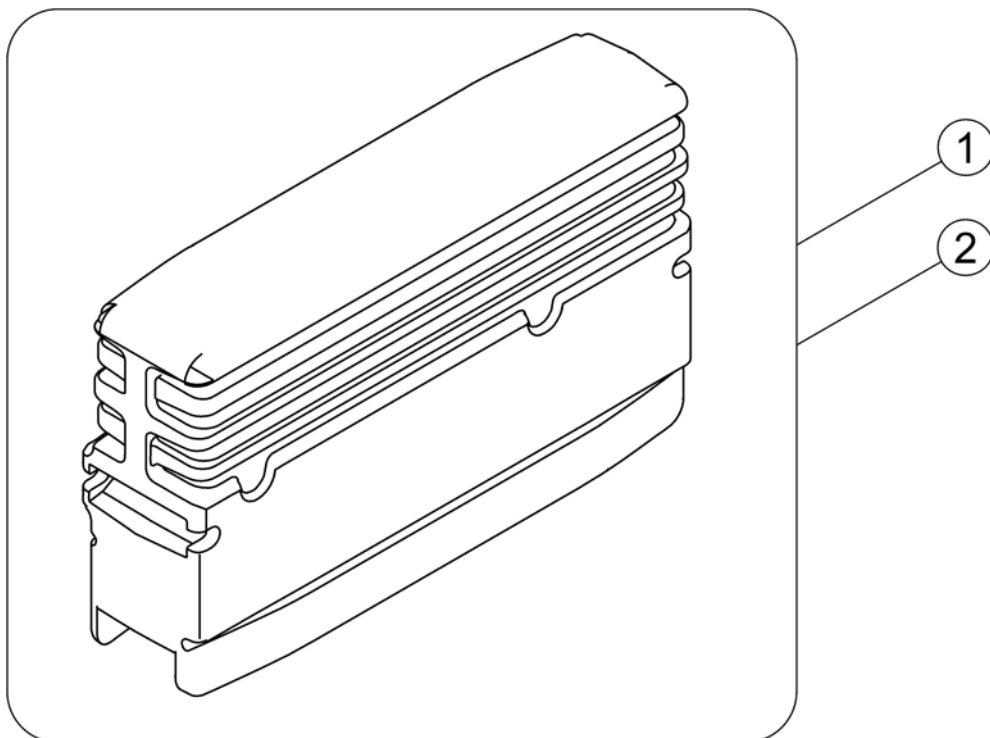


Pos.	Part number	Description	Valid from → until	Qty
1	SP1524244	Powermodule, ACS2, 90A, with light, DX2-PMA90L <i>Lock washer between module and bracket!</i>		1
2	SP1537975	Powermodule, ACS2, 90A, for light and G-Trac, DX2-PMA90LG <i>Lock washer between module and bracket!</i>		1



# Bracket for Power Modul Wide Base


1583336.iso

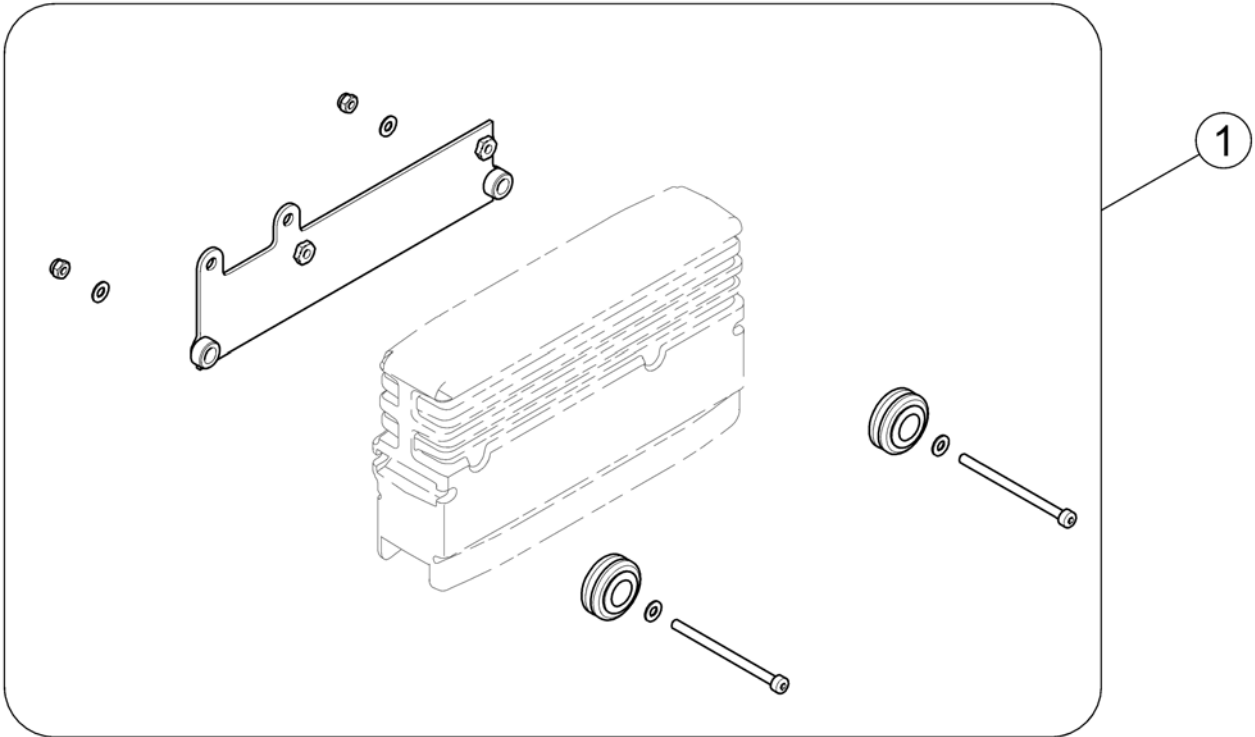

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583340	Bracket Power Module		1

## ACT for ACS 2


1528542X.iso

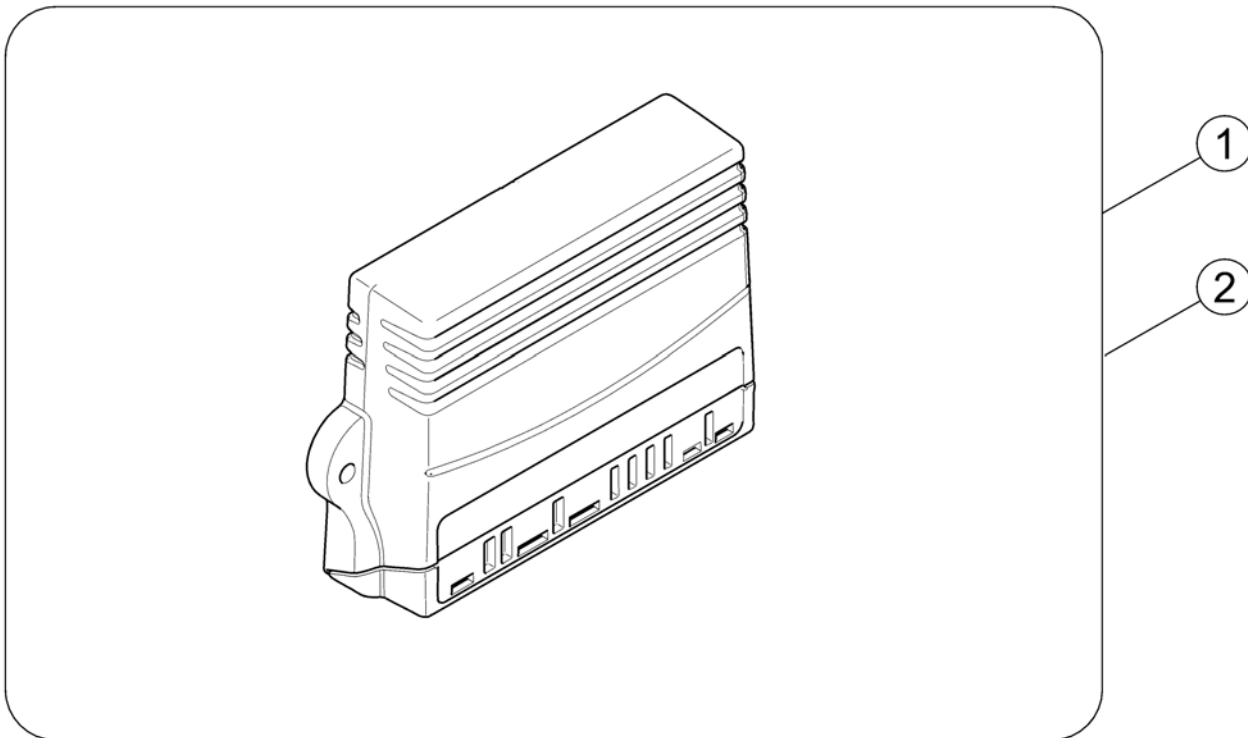

Pos.	Part number	Description	Valid from → until	Qty
1	SP1528542	ACT2 for ACS2, cpl.		1
2	SP1529382	ACT4 for ACS2, cpl.		1

## Bracket for ACT Wide Base


1563343 iso

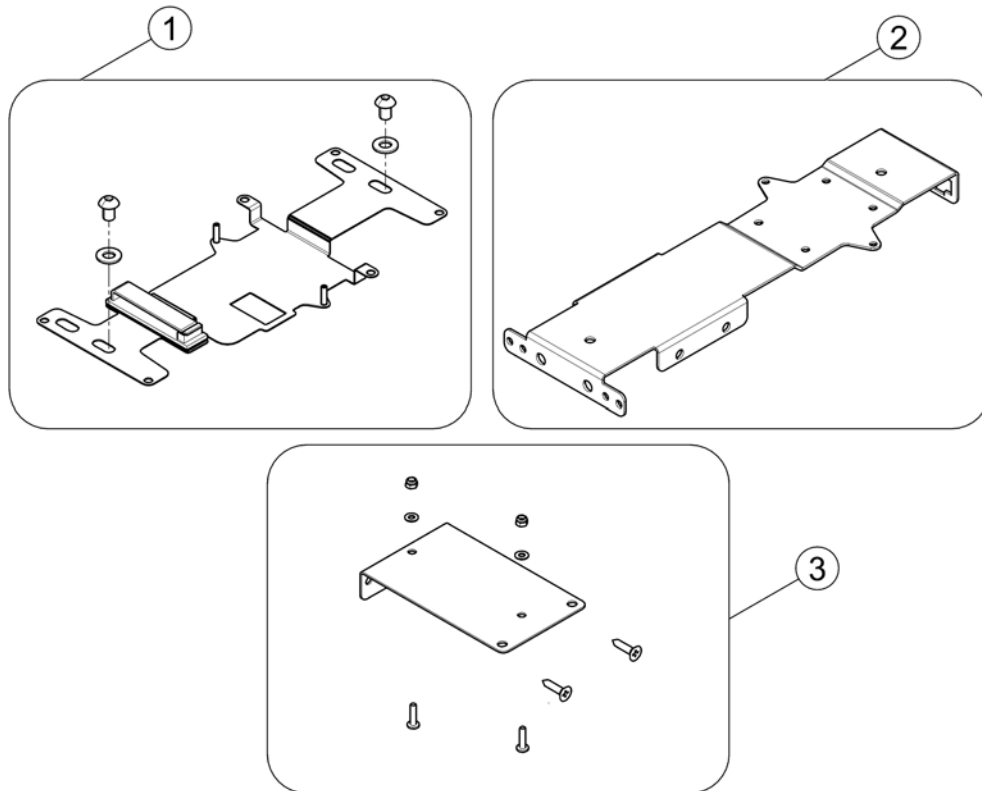

Pos.	Part number	Description	Valid from → until	Qty
1	SP1569960	ACT bracket		1
2	SP1564650	Multi support bracket		1
3	SP1580690	Bracket ACT Modul - Max Seat		1

# Light Board


1583345.iso

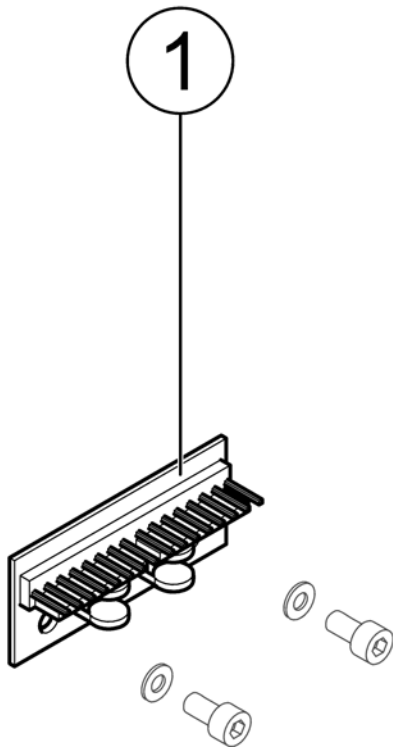

Pos.	Part number	Description	Valid from → until	Qty
1	1415004	Central Light Distributor Circuit Board ACS/Shark		1


# G-Trac-sensor

1537848X.dwg

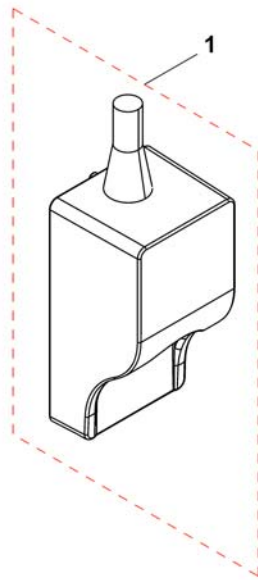

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541535	G-Trac		1


## ACS Remote & Brackets

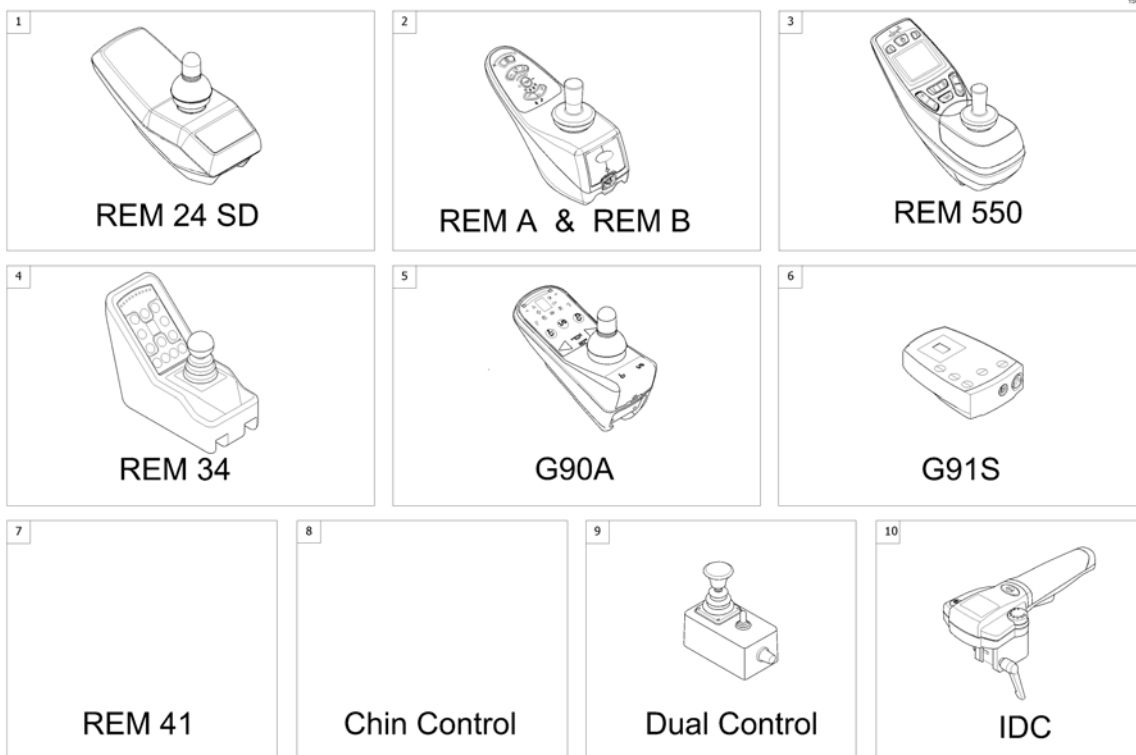

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24 SD with Brackets <i>[1601/1602]</i>		1
2	-	Remote REM A & B with Brackets <i>[1638/1639/1643/1644]</i>		1
3	-	Remote REM 550 with Brackets <i>[1617/1618]</i>		1
4	-	Remote REM 34 with Brackets <i>[1611]</i>		1
5	-	Remote G90A with Brackets <i>[1614/1616]</i>		1
6	-	Remote G91S with Brackets <i>[1615]</i>		1
7	-	Centre control ACS REM41 <i>[1607/1608]</i>		1
8	-	Remote Small Chin Control with Brackets <i>[1609]</i>		1
9	-	Remote Dual Control with Brackets <i>[1291/1292]</i>		1
10	-	Remote IDC Dual Control with Brackets <i>[1645]</i>		1

Invacare® TDX-SP2, TDX-SP2 NB | Electronic  
**Remote REM 24 SD with Brackets**


1562656 iso

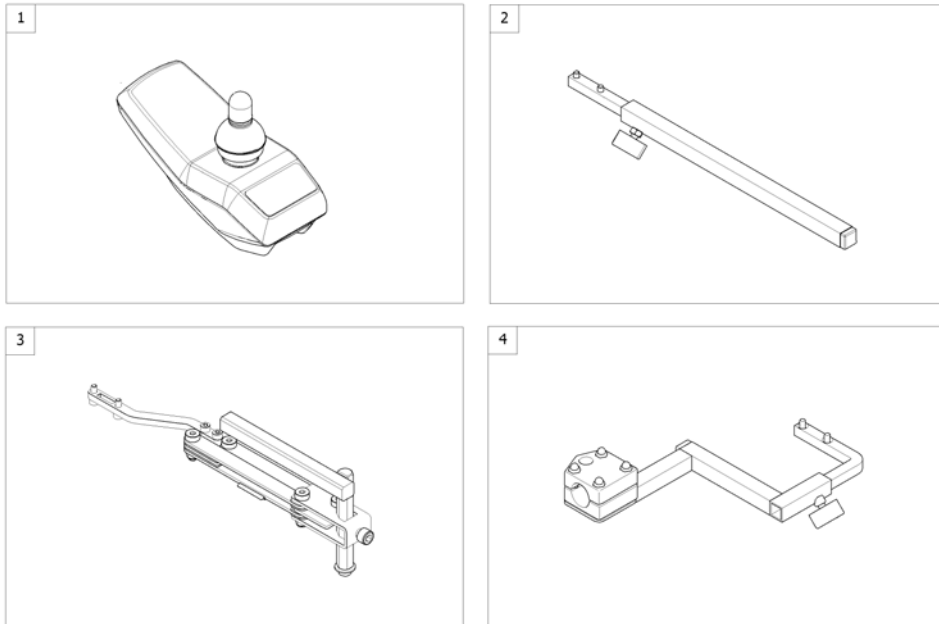

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24SD [1601/1602]		0
2	-	Remote Bracket Standard [1601+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1601+1707/1708]		1
4	-	Remote Bracket Attendant [1602+1207/1208]		1

# Remote REM 24SD

142802 860

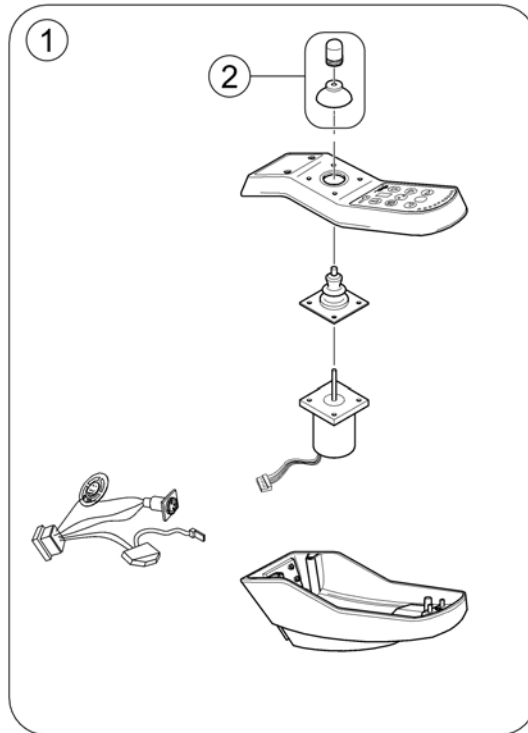


Pos.	Part number	Description	Valid from → until	Qty
1	1436541	Remote REM24SD		1
2	SPZ183229	Remote knob cpl.		1



# Remote Bracket Standard


1559851.iso

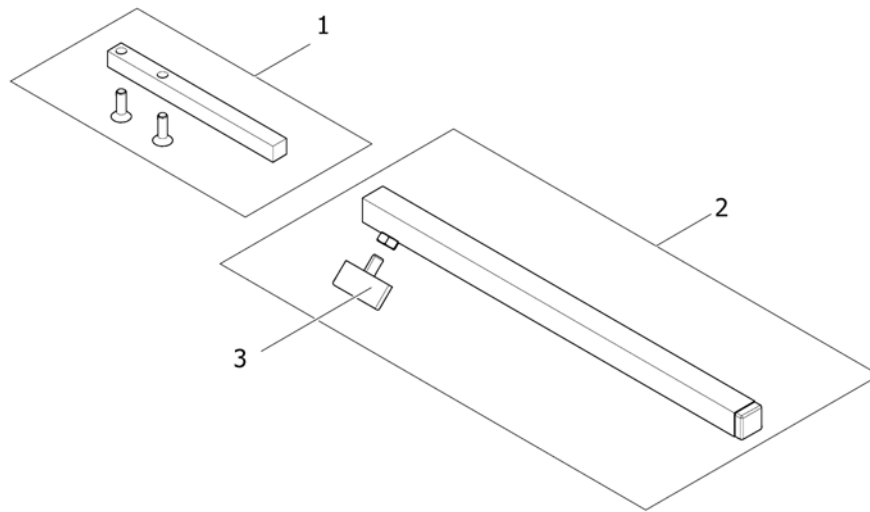

Pos.	Part number	Description	Valid from → until	Qty
1	SP1436542	Support, Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

## Remote Bracket Swing away & height adjust.


1559852.iso

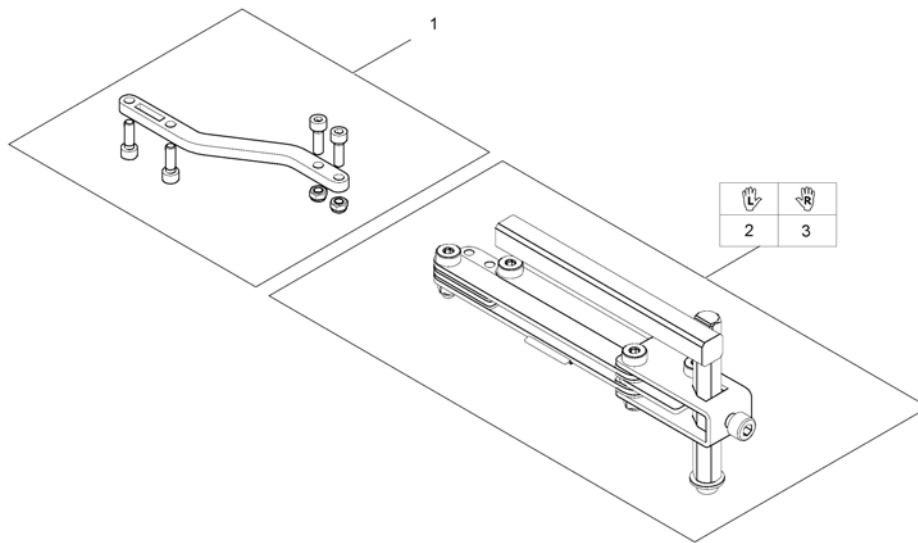

Pos.	Part number	Description	Valid from → until	Qty
1	SP1498222	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

# Remote Bracket Attendant


1560837.00

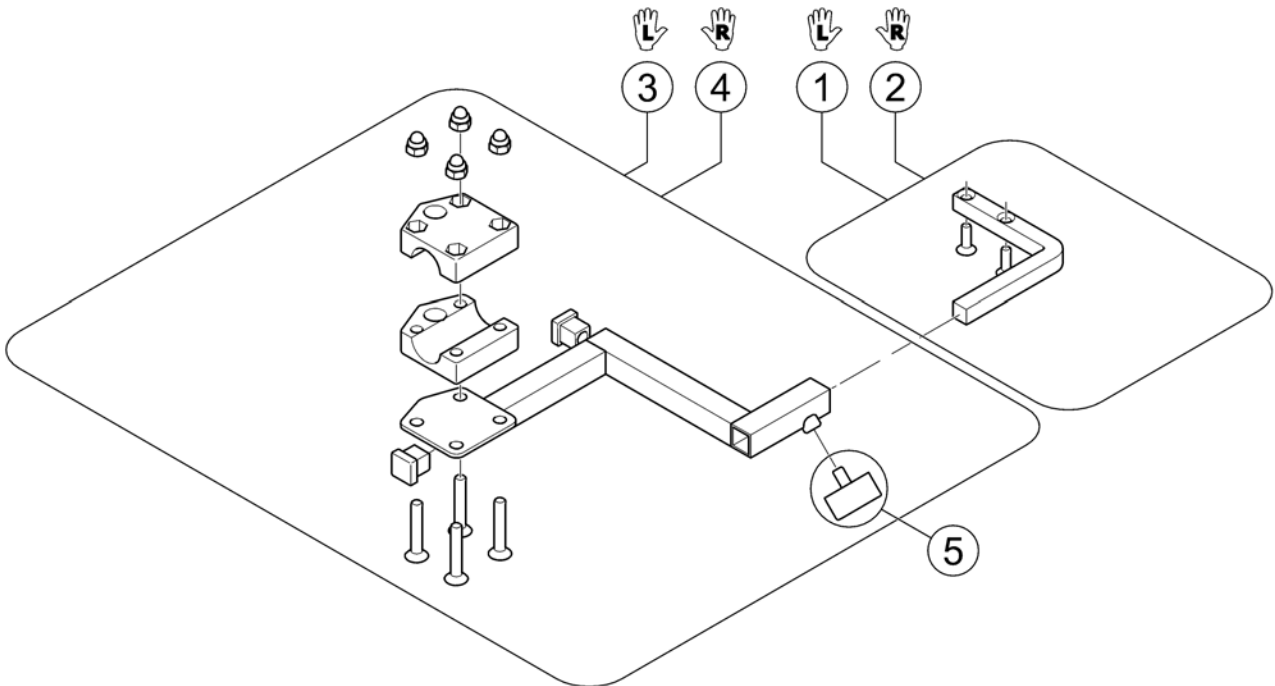

Pos.	Part number	Description	Valid from → until	Qty
1	1422806	Remote Support LH		1
2	1422810	Remote Support RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1

## Remote REM A & B with Brackets


1562687.iss

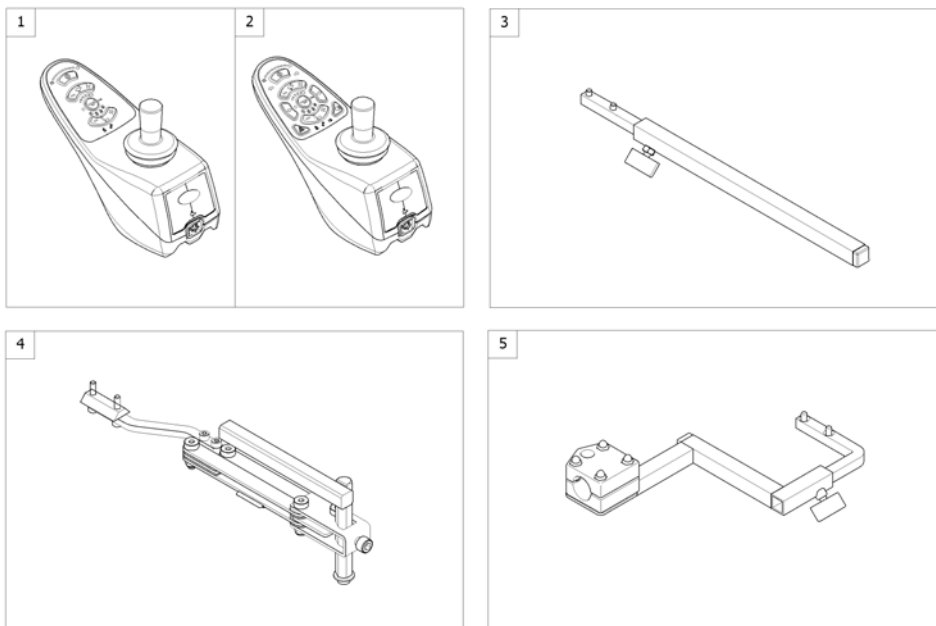

Pos.	Part number	Description	Valid from → until	Qty
1	SP1523440	Remote "REM A" [1638/1643]		1
2	SP1523445	Remote "REM B" [1639/1644]		1
3	-	Remote Bracket Standard [1638/1639+1207/1208]		1
4	-	Remote Bracket Swing away & height adjust. [1638/1639+1707/1708]		1
5	-	Remote Bracket Attendant [1643/1644+1207/1208]		1

# Remote Bracket Swing away & height adjust.


1559857.iso

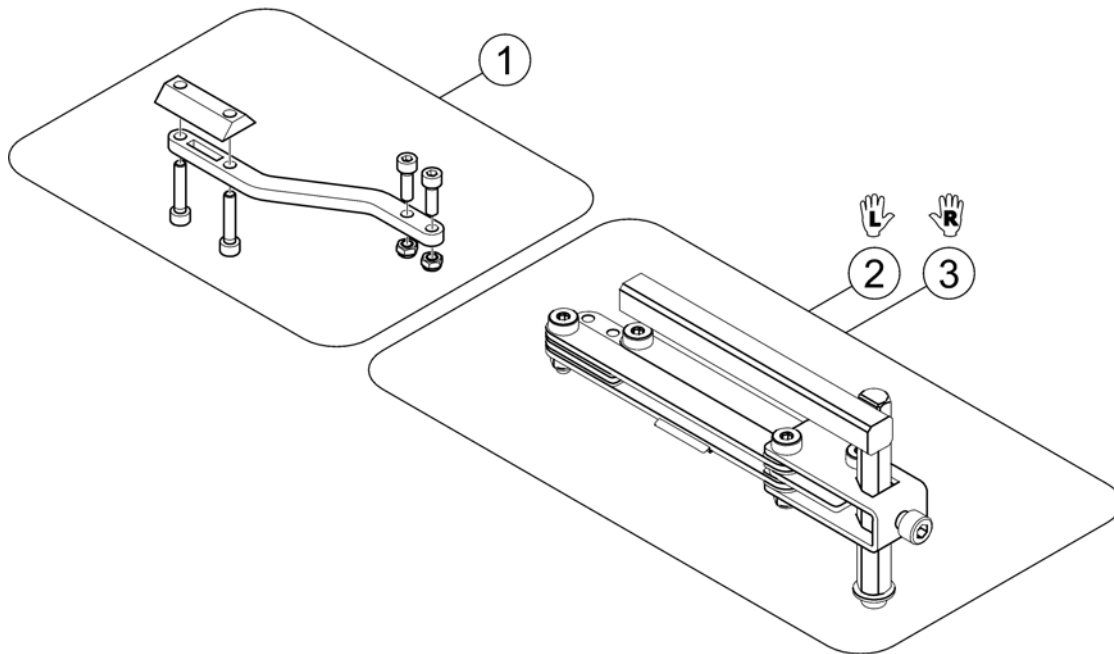



Pos.	Part number	Description	Valid from → until	Qty
1	SP1536180	Support for Remote Bracket, height adjustable & swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

## Remote REM 550 with Brackets

1562658 iso

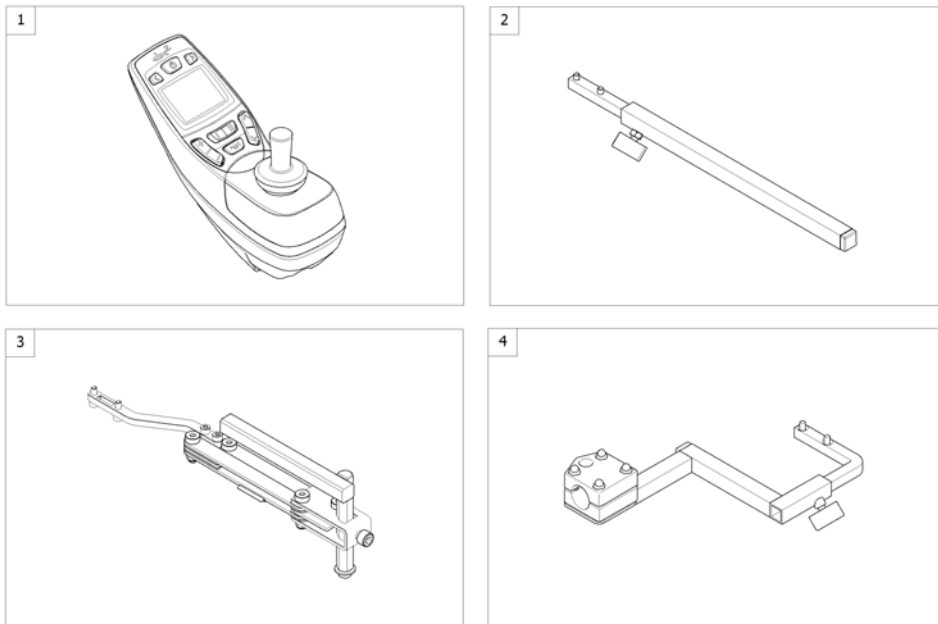


Pos.	Part number	Description	Valid from → until	Qty
1	SP1523444	Remote REM550 <i>[1617/1618]</i>		1
2	-	Remote Bracket Standard <i>[1617+1207/1208]</i>		1
3	-	Remote Bracket Swing away & height adjust. <i>[1617+1207/1208]</i>		1
4	-	Remote Bracket Attendant <i>[1618+1207/1208]</i>		1

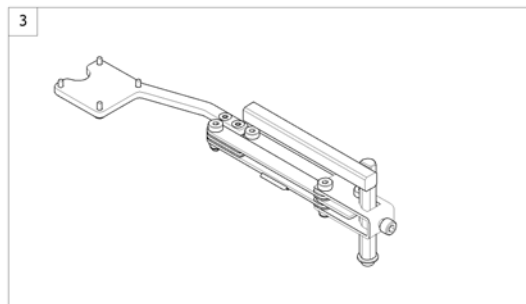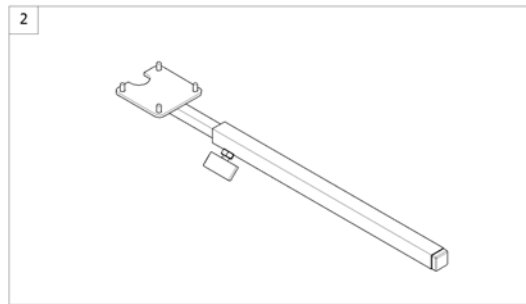

Pos.	Part number	Description	Valid from → until	Qty
1	SP1440677	Remote REM 34 [1611]		1
2	-	Remote Bracket Standard [1611+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1611+1707/1708]		1

# Remote Bracket Standard

1560844.iso

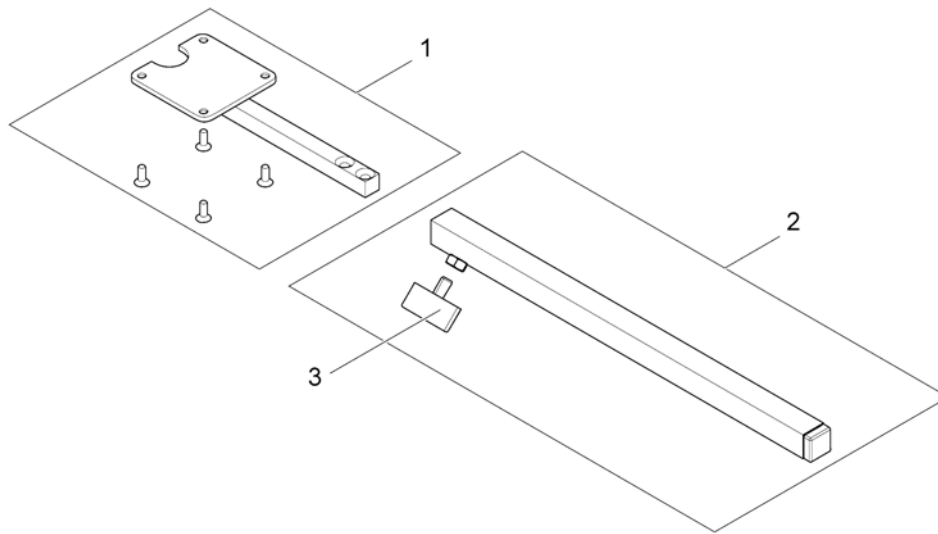


Pos.	Part number	Description	Valid from → until	Qty
1	SP1434921	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1



# Remote Bracket Swing away & height adjust.


1560845.iso

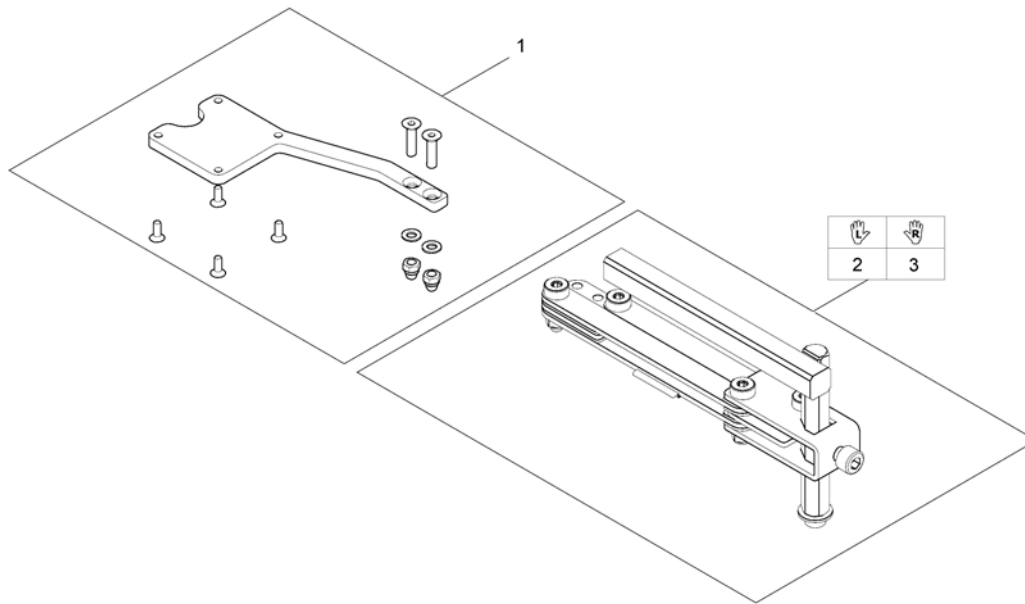

Pos.	Part number	Description	Valid from → until	Qty
1	SP1527125	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

## Remote G90A with Brackets


1562659.iso

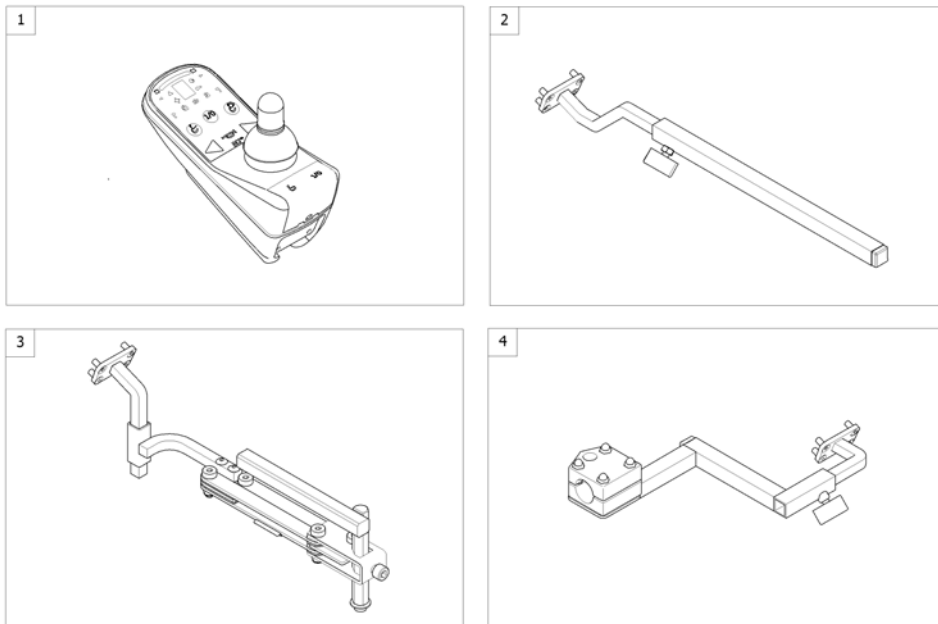

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504720	Remote G90A [1614/1616]		1
2	-	Remote Bracket Standard [1614+1207/1208]		1
3	-	Remote Bracket Swing away & height adjust. [1614+1707/1708]		1
4	-	Remote Bracket Attendant [1616+1207/1208]		1

# Remote Bracket Standard


1562939.iso

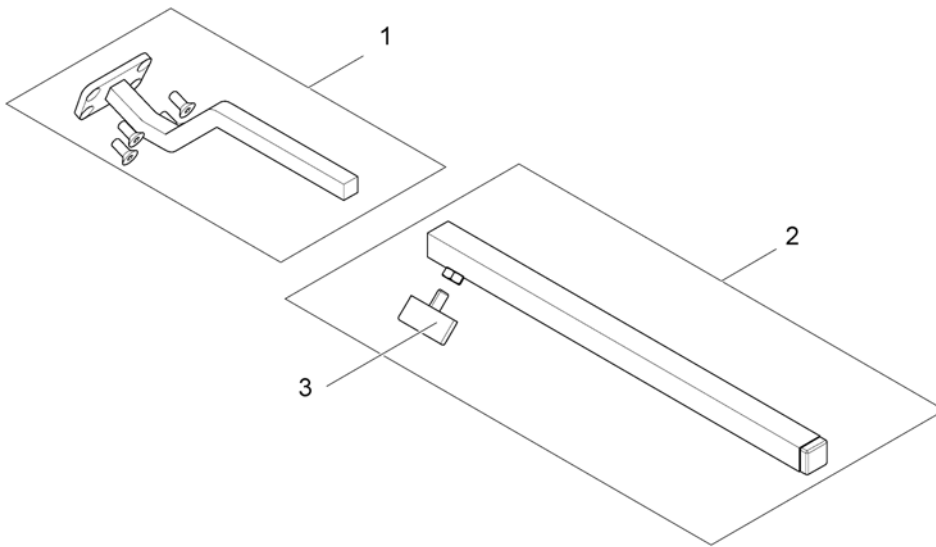

Pos.	Part number	Description	Valid from → until	Qty
1	SP1504822	Remote Support Standard		1
2	SPS183040	Remote Bracket, Standard		1
3	L31001	Wing ScREW M6X10		1

## Remote Bracket Swing away & height adjust.


1560310.iso

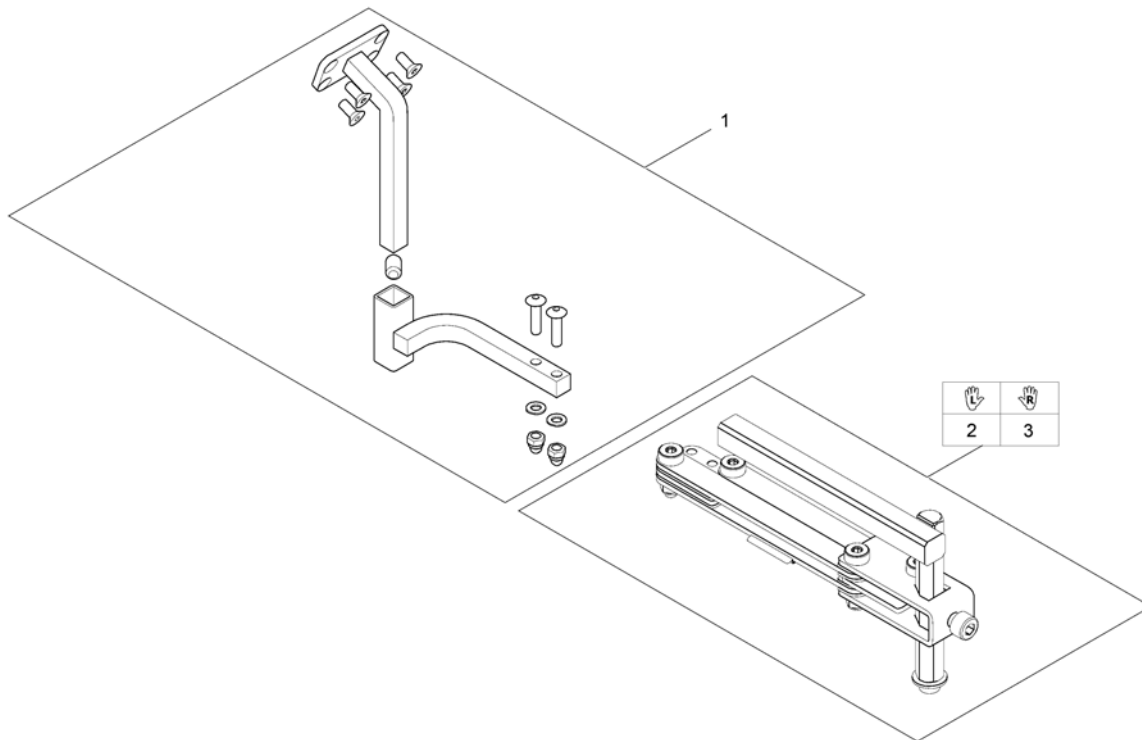

Pos.	Part number	Description	Valid from → until	Qty
1	SP1517352	Remote Support for Bracket height adjustable and swing away		1
2	SP1496249	Remote bracket, height adjustable & swing away, LH		1
3	SP1496248	Remote bracket, height adjustable & swing away, RH		1

# Remote Bracket Attendant



1560853.iso

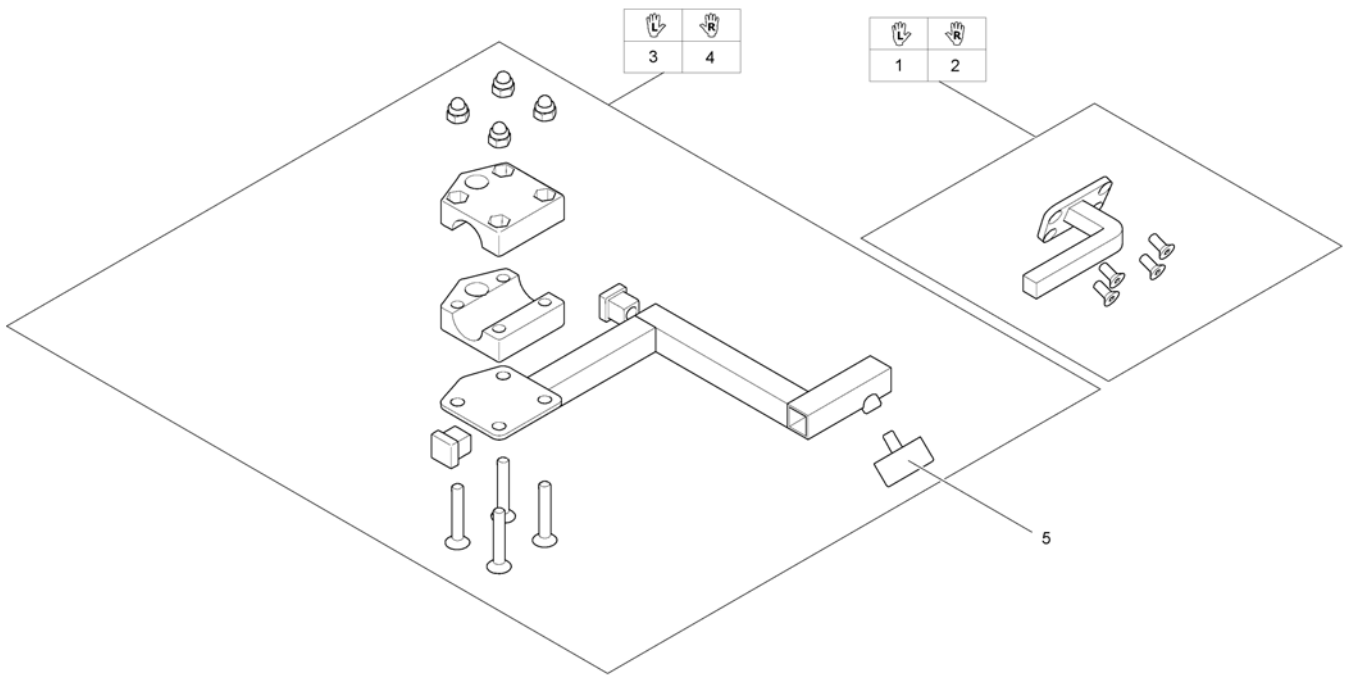

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509435	Remote Support attendant control G90A, LH		1
2	SP1509439	Remote Support attendant control G90A, RH		1
3	SP1422937	Bracket for attendant control LH		1
4	SP1422936	Bracket for attendant control RH		1
5	L31001	Wing ScREW M6X10		1

## Remote G91S with Brackets


1560850 iso

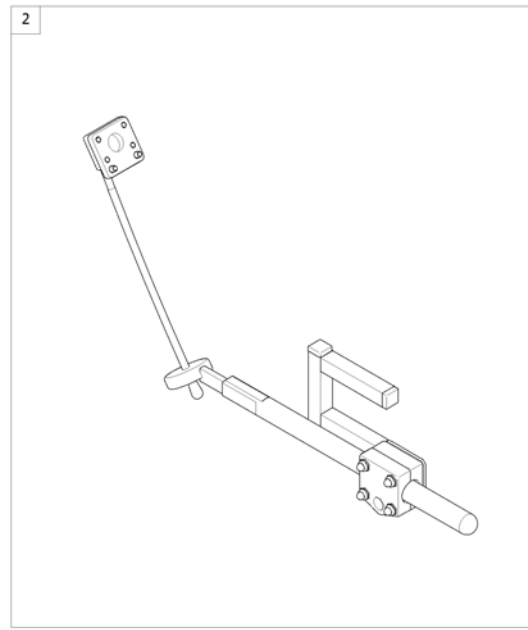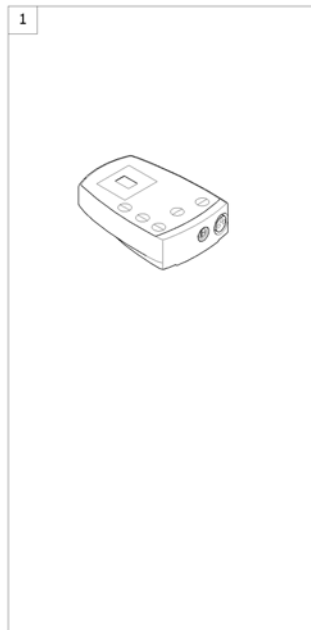

Pos.	Part number	Description	Valid from → until	Qty
1	SP1505187	Remote Control G91S [1615]		1
2	-	Remote Bracket Standard [1615+1207/1208]		1

# Remote Bracket Standard


1560654.iso

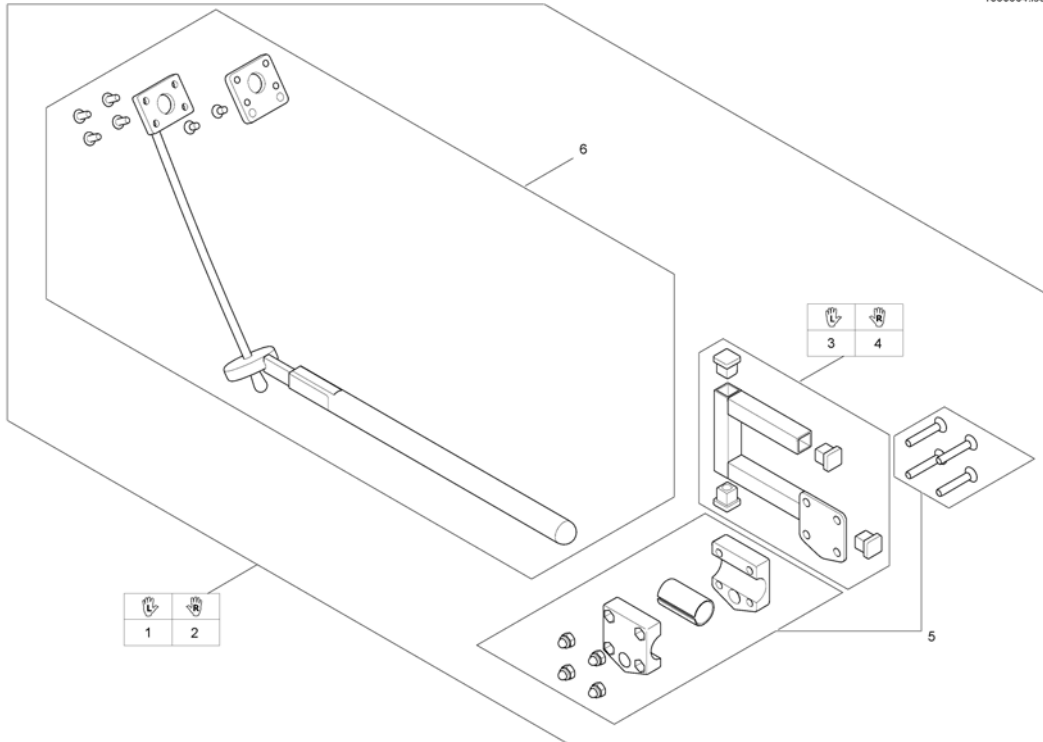

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509718	Remote Bracket G91/G91S LH, cpl		1
2	SP1509690	Remote Bracket G91/G91S RH, cpl.		1
3	SP1509716	Adapter G91 LH		1
4	SP1509711	Adapter G91 RH		1
5	SP1527503	Gripper clamp cpl.		1
6	SP1527504	Swan neck remote bracket with adapter plate		1

# Centre control ACS REM41

F383057.dwg

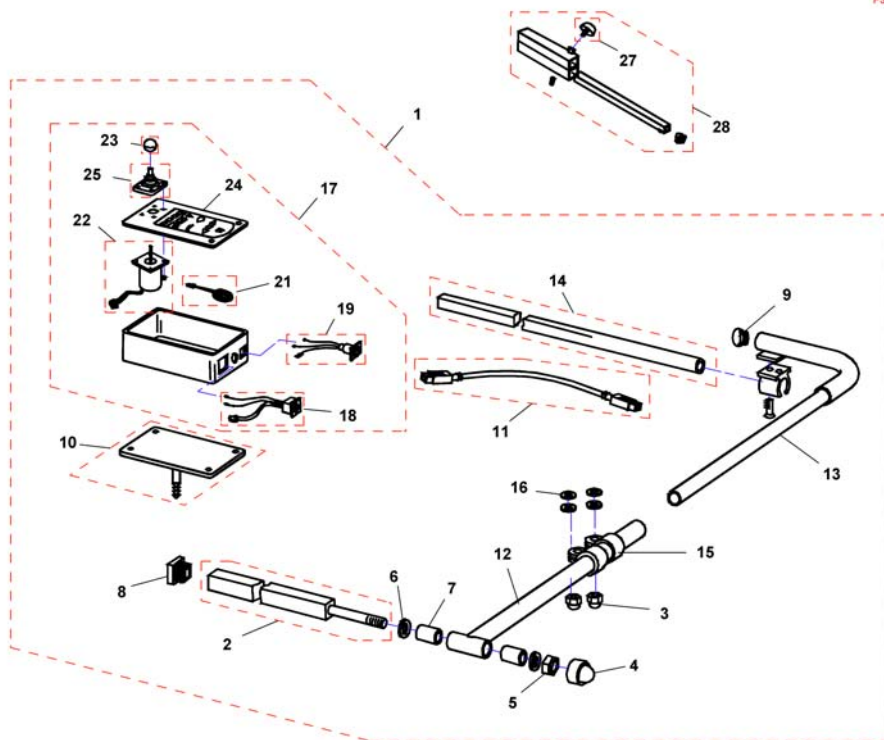


Pos.	Part number	Description	Valid from → until	Qty
-	Q99081	User Manual Table- & Middle Control REM41-S German		1
1	SP1530588	CENTRAL STEERING CPL.; SWING-AWAY RH, REM41-D		1
1	SP1530589	CENTRAL STEERING CPL.; SWING-AWAY LH, REM41-D		1
2	Z177024.00	SUPPORT TUBE; CHROME PLATED		1
3	F60208	Cap nut DIN1587-M08 zc		1
4	L01037	Sks Nut Cap M12		1
5	F20106	Hexagon nut DIN985-M12 zc		1
6	G00104	Washer DIN125-Ø13 zc		1
7	Z17032A	Bush Rd. 19 X 3.5 X 25		1
8	L01055	Plug 15x15mm		1
9	L01008	Tube End Plug SI 22		1
10	Z183216.P40	SUPPORT BRACKET; RAL 9005 BLACK		1
11	1559862	Bus cable ACS, 1700mm		1
12	Z183089.00	SWING ARM; CHROME PLATED		1
13	Z183086.00	SUPPORT ARM RH; CHROME PLATED		1
13	Z183092.00	SUPPORT ARM LH; CHROME PLATED		1
14	H281005A.00	MIRROR SUPPORT RAIL; CHROME PLATED		1
15	5006720	Klemmschelle F. Dm 22 mm		1
16	G00106	Washer DIN125-A-Ø8,4 zc		1
17	SP1527930	DX ACS Tray Remote DX-REM41		1
18	P41108	ACS-BUS CABLE SOCKET		1
19	P41106	CHARGER SOCKET		1
21	P30493	HORN		1
22	GSM55120	ACS-remote WITH 180MM CABLE		1
23	L36999	KNOB FOR remote 32 MM, HOLE 5.8 MM		1
24	-	No longer available.		0
25	Z183001A	Rubber gaiter for ACS remote		1
26	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1
27	L31001	Wing ScREW M6X10		1
28	Z183289	CENTRAL STEERING SUPPORT TUBE CPL.		1



## Remote Small Chin Control with Brackets


Pos.	Part number	Description	Valid from → until	Qty
-		Small Chincontrol		1
-		Framework for chincontrol		1

## Small Chincontrol

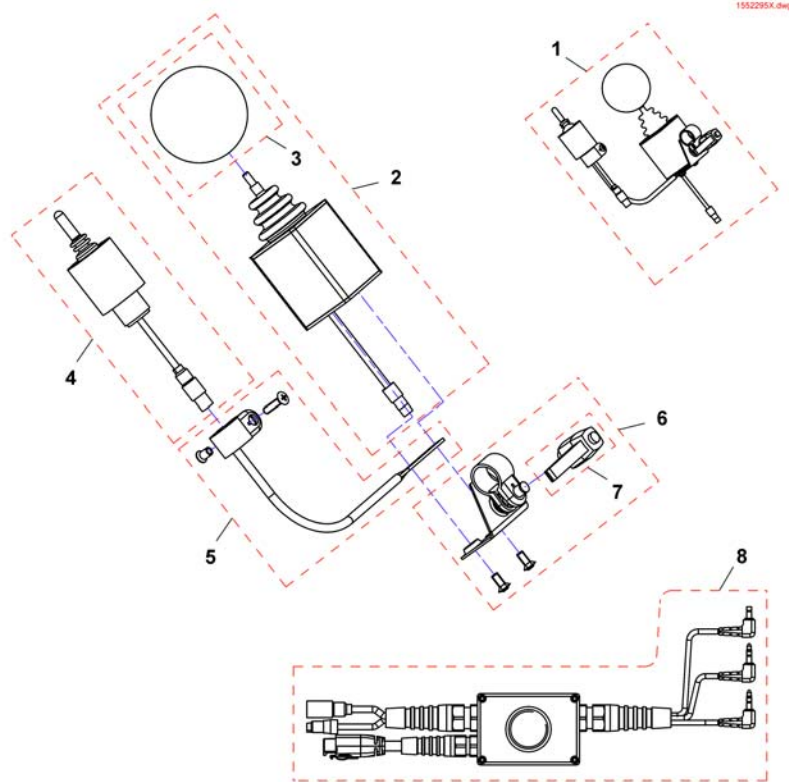

Pos.	Part number	Description	Valid from → until	Qty
1	1448705	Chin control REM24, cpl.		1
1	1421770	Chin control G80/G90, cpl.		1
1	SP1544255	Chin control G91, cpl.		1
2	Z183282	Joystick, kmpl.		1
3	Z183265	Foam control knob cpl.		1
4	1442943	Switchjoystick REM24, kmpl.		1
4	Z183283	Switchjoystick G80/G90, cpl.		1
4	SP1544250	Switchjoystick G91, kmpl.		1
5	SP1552294	Switchjoystick bracket		1
6	SP1552295	Joystick clamp		1
7	L32901	Clampinglever		1
8	SP1442944	Harness REM24		1
8	Z183275	Harness G80/G90		1
8	SP1541715	Harness G91		1




## Framework for chincontrol

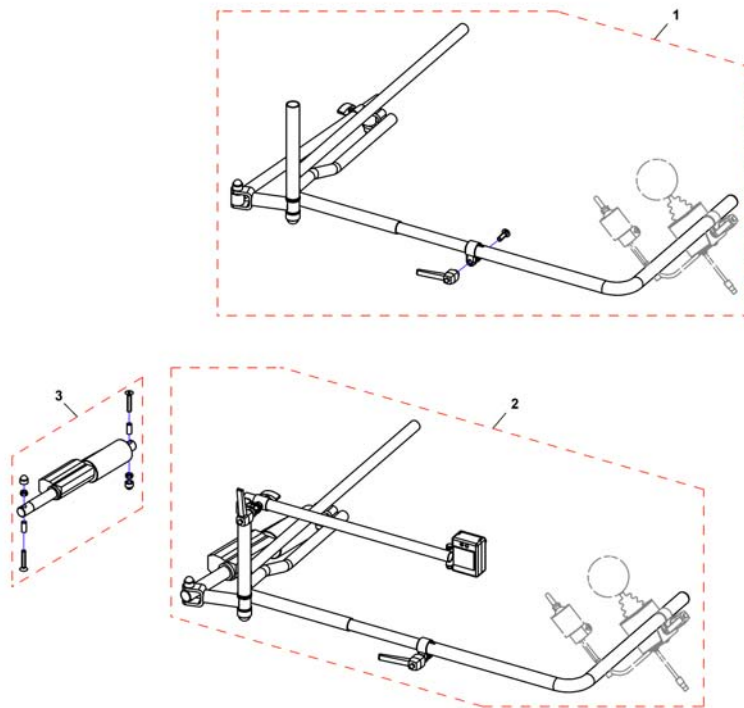

Pos.	Part number	Description	Valid from → until	Qty
-	Z167994	CHIN CONTROL SUPPORT . CPL. <i>Not shown / Nicht abgebildet / Non illustré</i>		1
1	SP1422943	Framework for chin control, manual, horizontal swing away, LH, cpl.		1
1	SP1422942	Framework for chin control, manual, horizontal swing away, RH, cpl.		1
2	SP1422945	Framework for chincontrol, powered, horizontal swing away, LH, cpl. <i>incl. Actuator</i>		1
2	SP1422944	Framework for chin control, powered, horizontal swing away, RH, cpl. <i>incl. Actuator</i>		1
-	SP1552335	Hardware, chin control, powered, horizontal swing away, LH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1552336	Hardware, chin control, powered, horizontal swing away, RH <i>Not shown / Nicht abgebildet / Non illustré</i>		1
-	SP1422946	Framework for chincontrol, powered, vertikal swing away <i>Not shown / Nicht abgebildet / Non illustré</i>		1
3	SP1552337	Actuator for horizontal swing away chin control, cpl.		1
3	SP1552418	Gestell für Kinnbedienung, elektrisch, vertikal abschwenkbar <i>Not shown / Nicht abgebildet / Non illustré</i>		1

# Remote Dual Control with Brackets


1574292 00

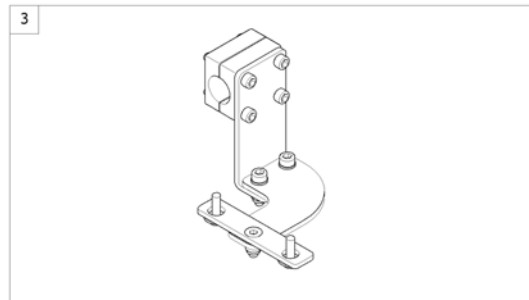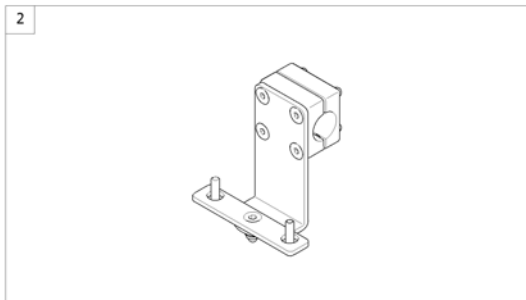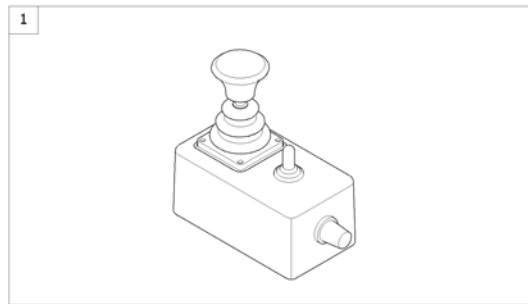

Pos.	Part number	Description	Valid from → until	Qty
1	6001425	ACS dual control <i>[1291/1292]</i>		1
2	-	Remote Bracket Dual Control for Max Seat		1
3	-	Remote Bracket Dual Control for Modulite Seat		1

# Remote Bracket Dual Control for Max Seat


1560885.iso

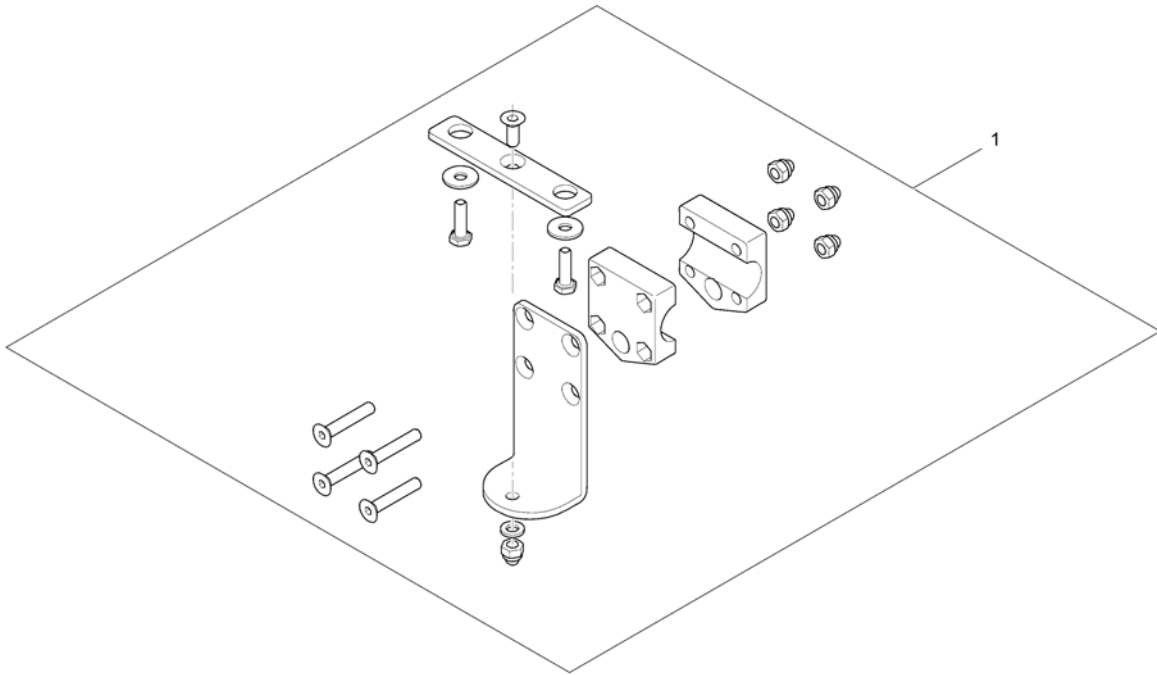

Pos.	Part number	Description	Valid from → until	Qty
1	SP1422818	Remote Bracket Dual Control		1

# Remote Bracket Dual Control for Modulite Seat


1574291 iso

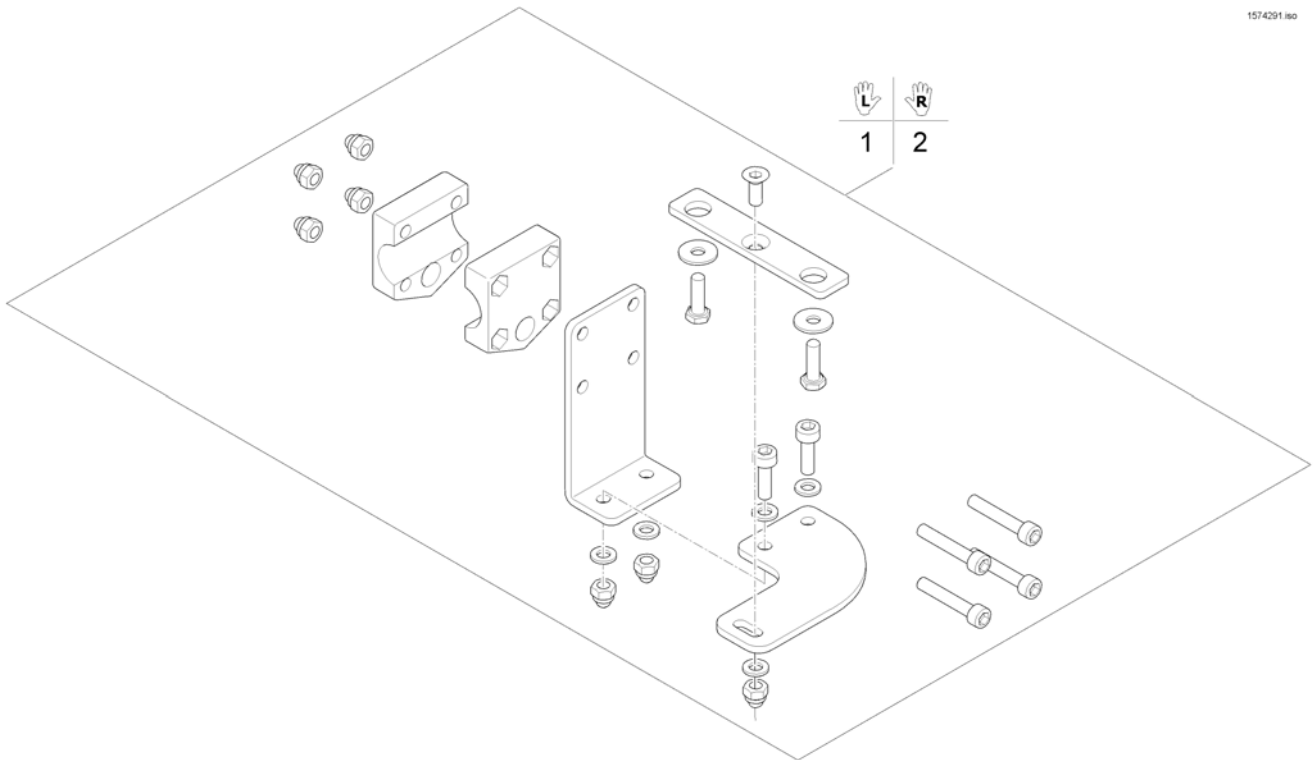

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571799	Remote Bracket Dual Control LH		1
2	SP1571800	Remote Bracket Dual Control RH		1

# Remote IDC Dual Control with Brackets

1509875X.iso

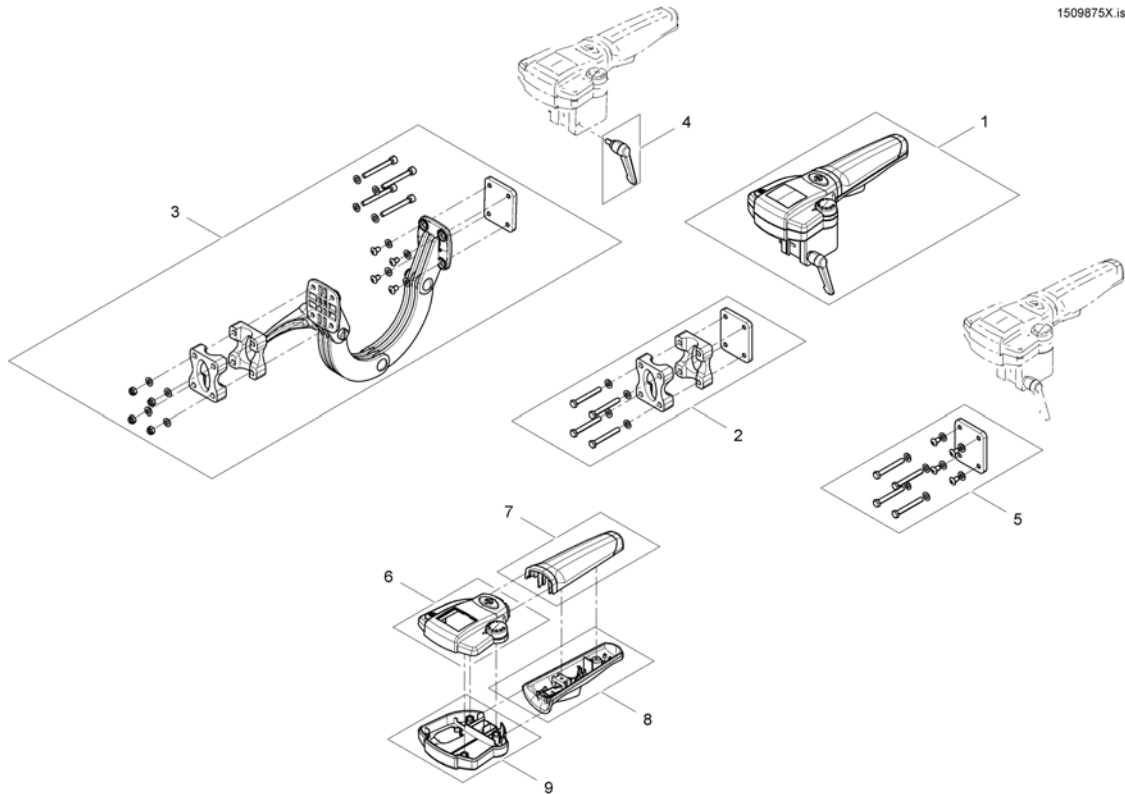

Pos.	Part number	Description	Valid from → until	Qty
1	SP1509875	Intuitiv Dual Control <i>For chairs build before 01/02/10 please also order SP1537148</i>		1
2	SP1522310	Fixed bracket		1
3	SP1521488	Adjustable bracket		1
4	SP1510135	Clamp lever M5x10		1
5	SP1537148	Supportplate IDC for chairs before 01.02.10		1
6	SP1579573	Upper Housing		1
7	SP1511060	Handhold upper part		1
8	SP1511099	Handhold lower part		1
9	SP1510470	Housing lower part		1


## ACS Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables ACS		0
-		Light Cable ACS		0
-		T-Adapter - Modulite & Max Seat		0
-		Programming Kits		0
-		Magnetkey for Remotes		0



# Bus-Cables ACS

ACSBUSKABEL.iso

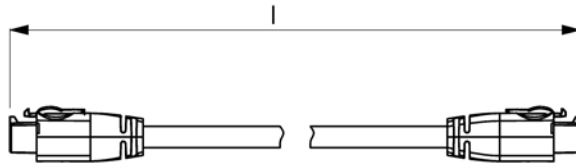


Position	length l [mm]
1	300
2	500
3	700
4	1100
5	1700
6	2000
7	2500
8	2700
9	1060


Pos.	Part number	Description	Valid from → until	Qty
1	GSM63003	Bus cable ACS, 300mm		1
2	SPGSM63005	Bus cable ACS, 500mm <i>Replace GSM63004</i>		1
3	GSM63007	Bus cable ACS 700mm		1
4	GSM63011	Bus cable ACS, 1100mm		1
5	1559862	Bus cable ACS, 1700mm <i>Replace GSM63016</i>		1
6	GSM63020	Bus cable ACS, 2000mm <i>Replace 1419386</i>		1
7	1440424	Bus cable ACS, 2500mm		1
8	SP1559873	Bus cable ACS, 2700mm <i>Replace GSM63026</i>		1
9	SP1550402	Extension cable ACS, 1060mm		1

# Light Cable ACS

1583704.00

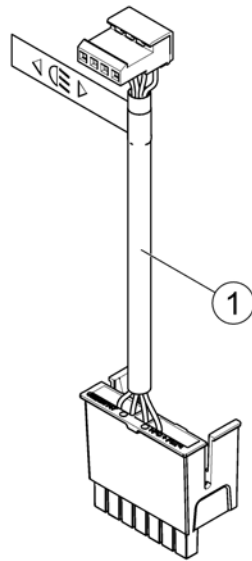


Pos.	Part number	Description	Valid from → until	Qty
1	SP1524237	Light cable ACS2		1

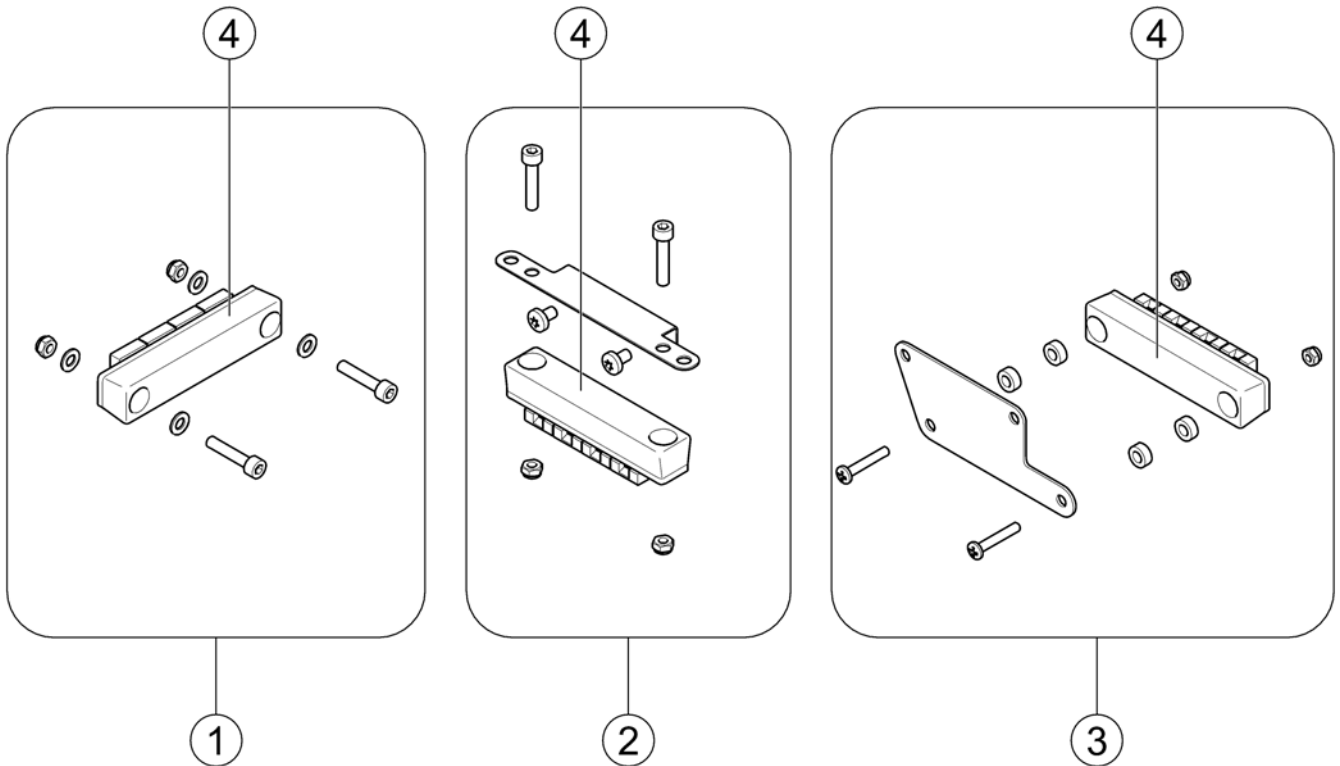

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571826	T-Adapter - Modulite Seat		1
2	SP1569958	T-Adapter - Modulite Seat with MBL-Modul		1
3	SP1580691	T-Adapter - Max Seat		1
4	P10072	ACS-Bus T-Adapter (Splitter)		1

# Programming Kits


1584805 iso

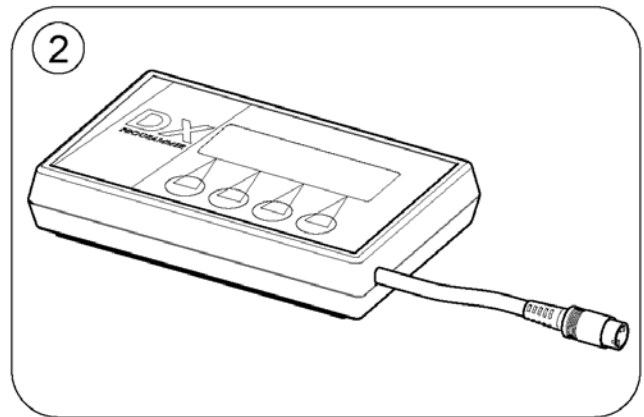

Pos.	Part number	Description	Valid from → until	Qty
1	1449280	USW wizard dongle kit, EDL-USW, cpl.		1
2	6001450	Hand Programmer, Instrucions In French, German, Spanish		1
	SP1441551	Programming Adapter Shark DK-ADAPT (necessary to program Shark)		1

# Magnetkey for Remotes

1564186.iso

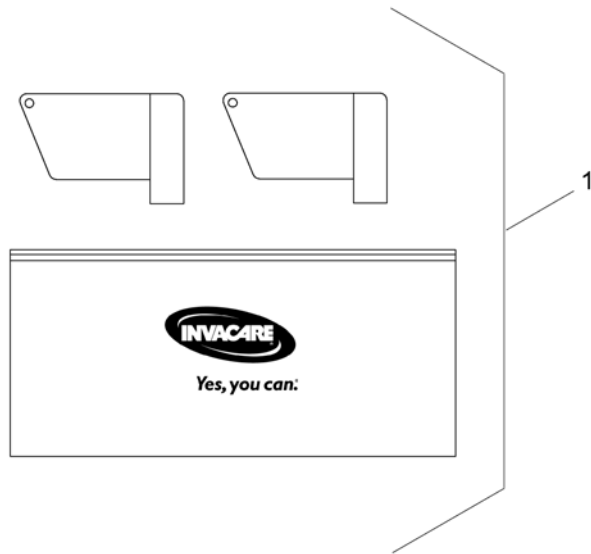

Pos.	Part number	Description	Valid from → until	Qty
1	SP1419695	Magnet key with bag		1

# Electronic for Narrow Base

SPCTDX\_SP70.iss



Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic		1
2	-	Shark Electronic		1

1500784.sp

1  
**Modules**

2  
**Remote  
 & Brackets**

3  
**Cables &  
 Accessories**

Pos.	Part number	Description	Valid from → until	Qty
1	-	ACS Electronic Modules		1
2	-	ACS Remote & Brackets		1
3	-	ACS Cables & Accessories		1


## ACS Electronic Modules

Pos.	Part number	Description	Valid from → until	Qty
-		Power Modul - ACS		1
-		Bracket for Power Modul Narrow Base		1
-		ACT for ACS 2		1
-		Bracket for ACT Narrow Base		1
-		Light Board		1
-		G-Trac-sensor		1



# Bracket for Power Modul Narrow Base


1583338.iso

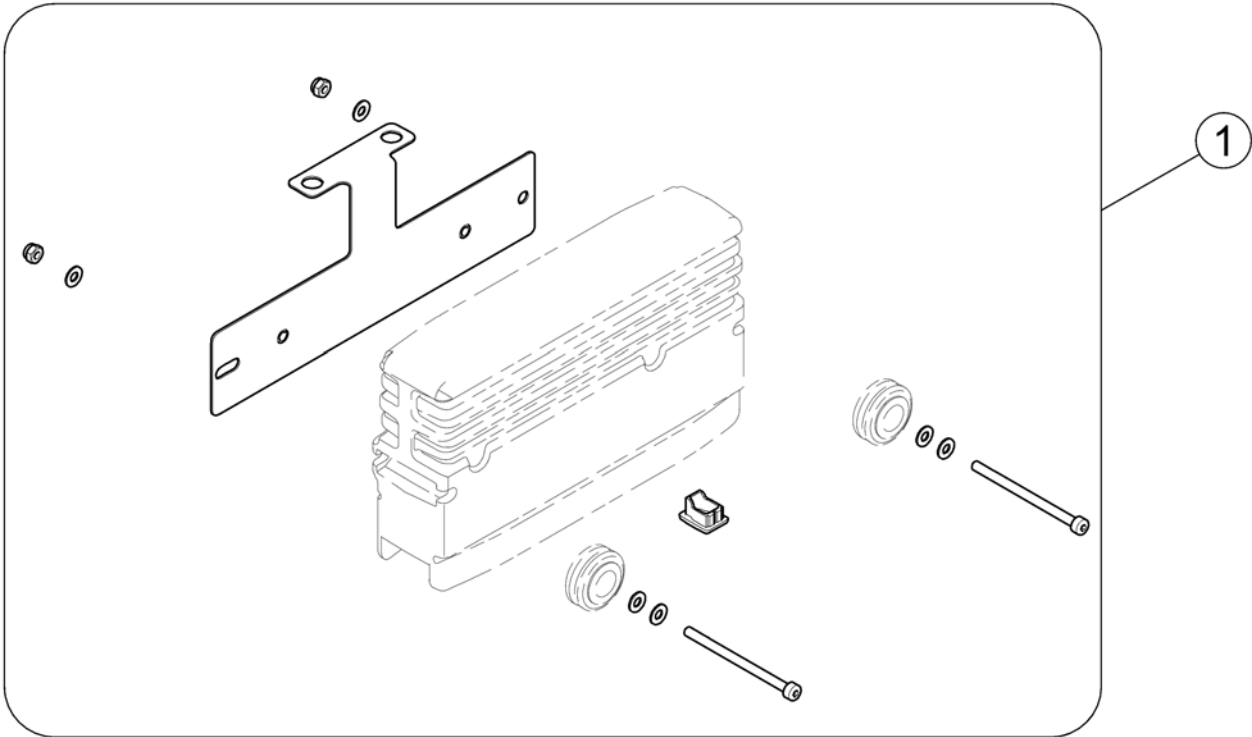

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583339	Bracket Power Modul		1

## Bracket for ACT Narrow Base


1583341.iso

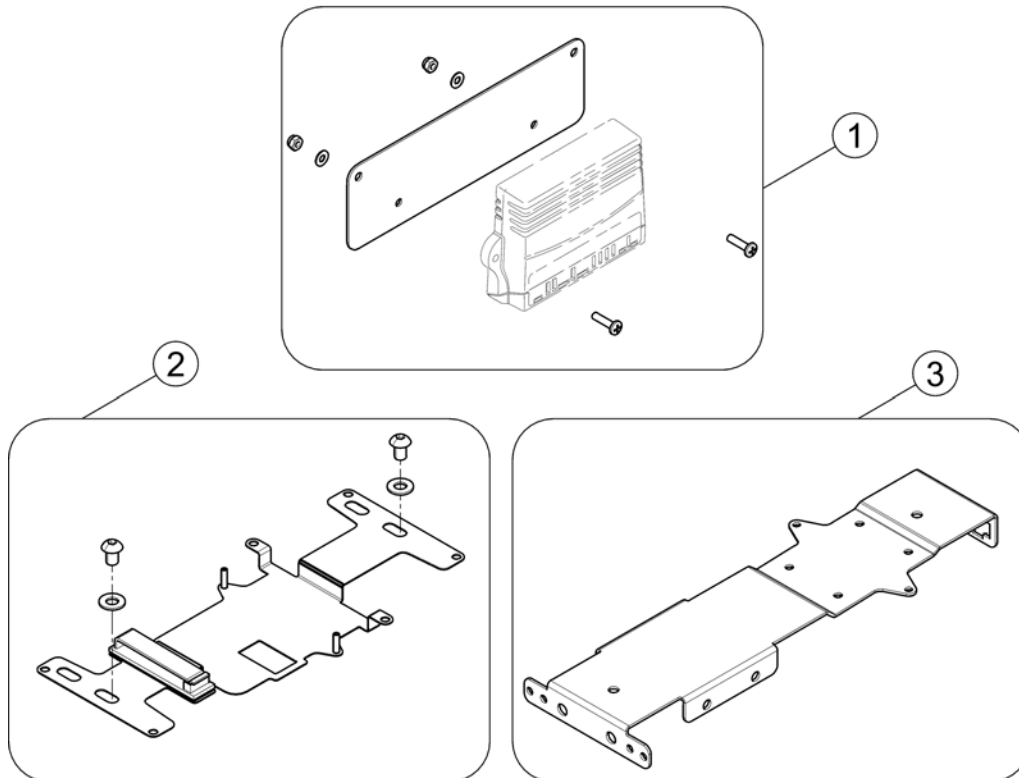

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583342	Bracket for ACT on Chassis (Narrow Base)		1
2	SP1569960	ACT bracket		1
3	SP1564650	Multi support bracket		1


# Light Board

1583344 iso

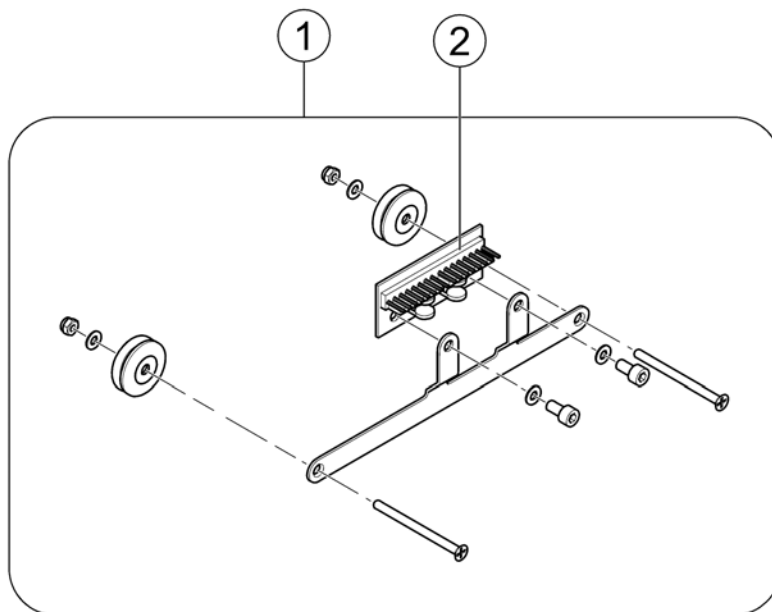



Pos.	Part number	Description	Valid from → until	Qty
1	SP1574115	Light board with bracket		1
2	1415004	Central Light Distributor Circuit Board ACS/Shark		1

## ACS Remote & Brackets

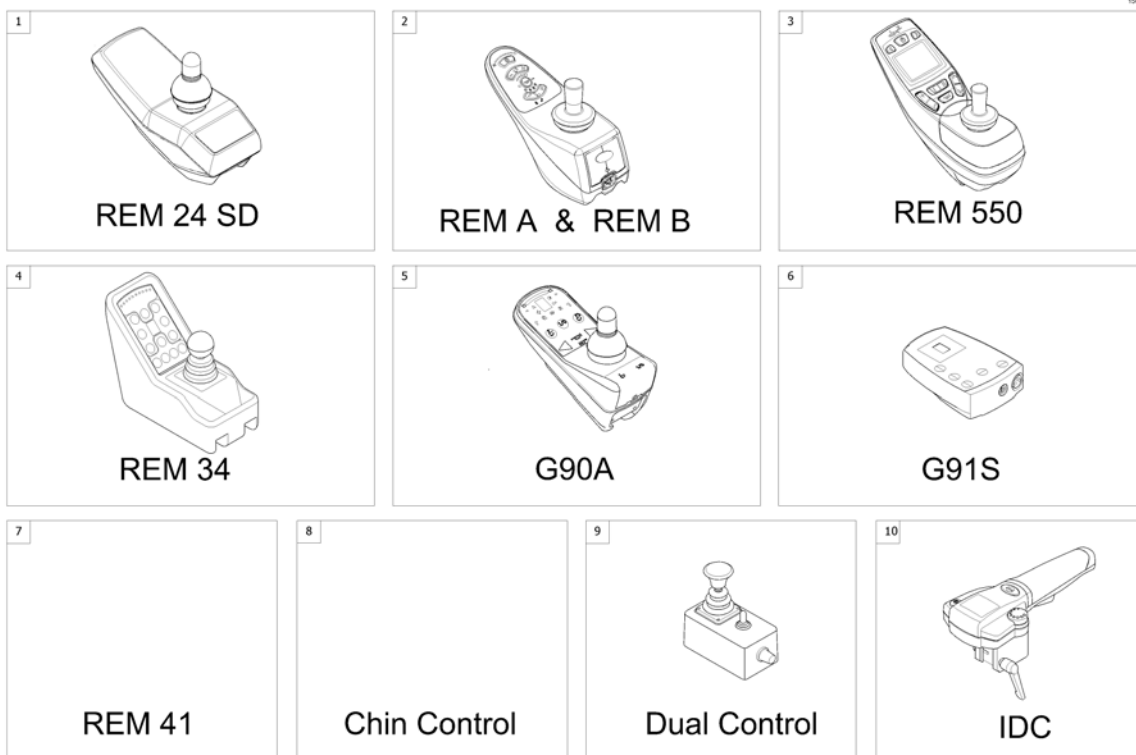

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote REM 24 SD with Brackets <i>[1601/1602]</i>		1
2	-	Remote REM A & B with Brackets <i>[1638/1639/1643/1644]</i>		1
3	-	Remote REM 550 with Brackets <i>[1617/1618]</i>		1
4	-	Remote REM 34 with Brackets <i>[1611]</i>		1
5	-	Remote G90A with Brackets <i>[1614/1616]</i>		1
6	-	Remote G91S with Brackets <i>[1615]</i>		1
7	-	Centre control ACS REM41 <i>[1607/1608]</i>		1
8	-	Remote Small Chin Control with Brackets <i>[1609]</i>		1
9	-	Remote Dual Control with Brackets <i>[1291/1292]</i>		1
10	-	Remote IDC Dual Control with Brackets <i>[1645]</i>		1

1583353.no

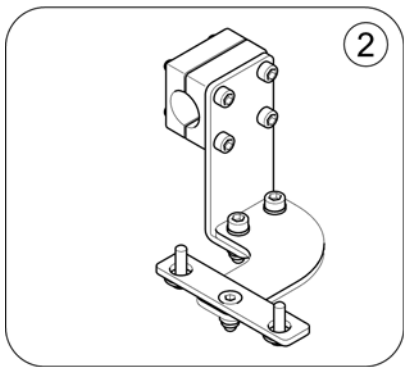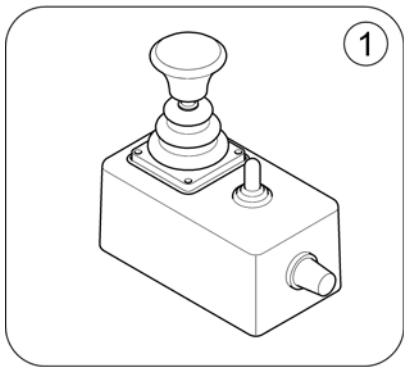


Pos.	Part number	Description	Valid from → until	Qty
1	6001425	ACS dual control [1291/1292]		1
2	-	Remote Bracket Dual Control for Modulite Seat		1

## ACS Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables ACS		0
-		Light Cable ACS		0
-		T-Adapter - Modulite Seat		0
-		Programming Kits		0
-		Magnetkey for Remotes		0

# T-Adapter - Modulite Seat


1583360 iso

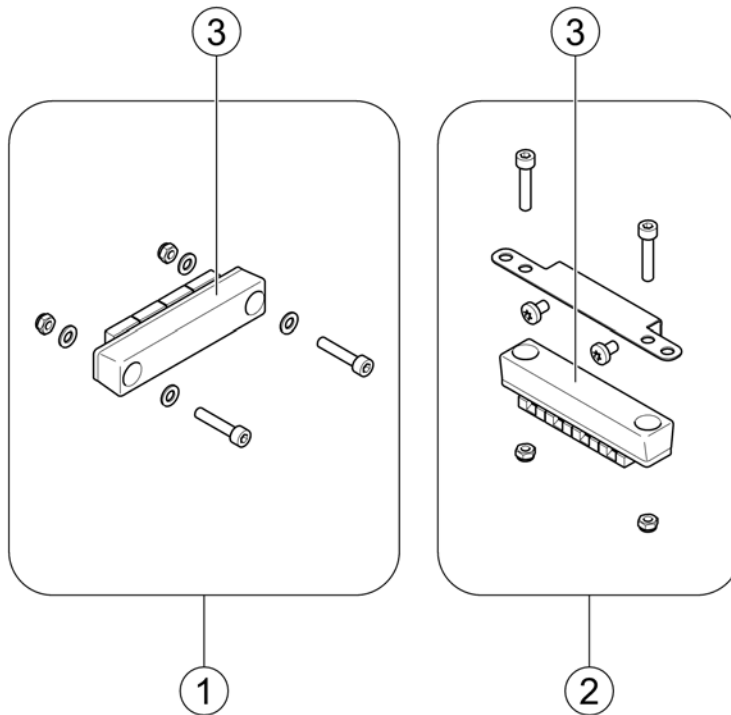

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571826	T-Adapter - Modulite Seat		1
2	SP1569958	T-Adapter - Modulite Seat with MBL-Modul		1
3	P10072	ACS-Bus T-Adapter (Splitter)		1

## Shark Electronic

1500784.sp

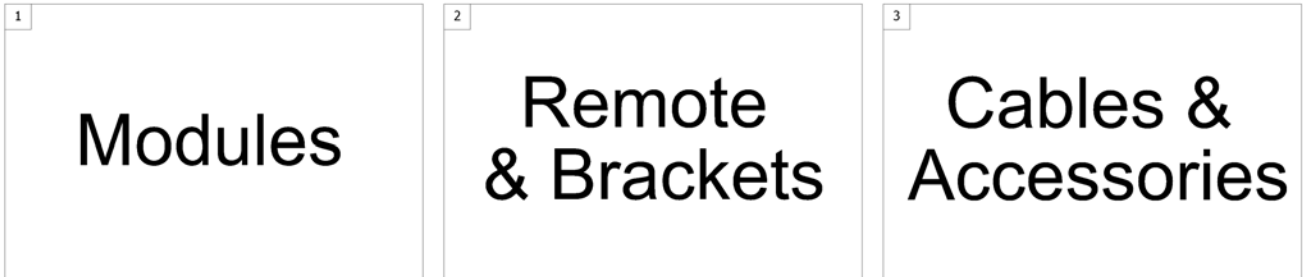

Pos.	Part number	Description	Valid from → until	Qty
1	-	Shark Electronic Modules		1
2	-	Shark Remote & Brackets		1
3	-	Shark Cables & Accessories		1




**Shark Electronic Modules**

Pos.	Part number	Description	Valid from → until	Qty
-		Power Modul		1
-		Bracket for Power Modul		1
-		Light Board		1

# Power Modul


1583356.iso

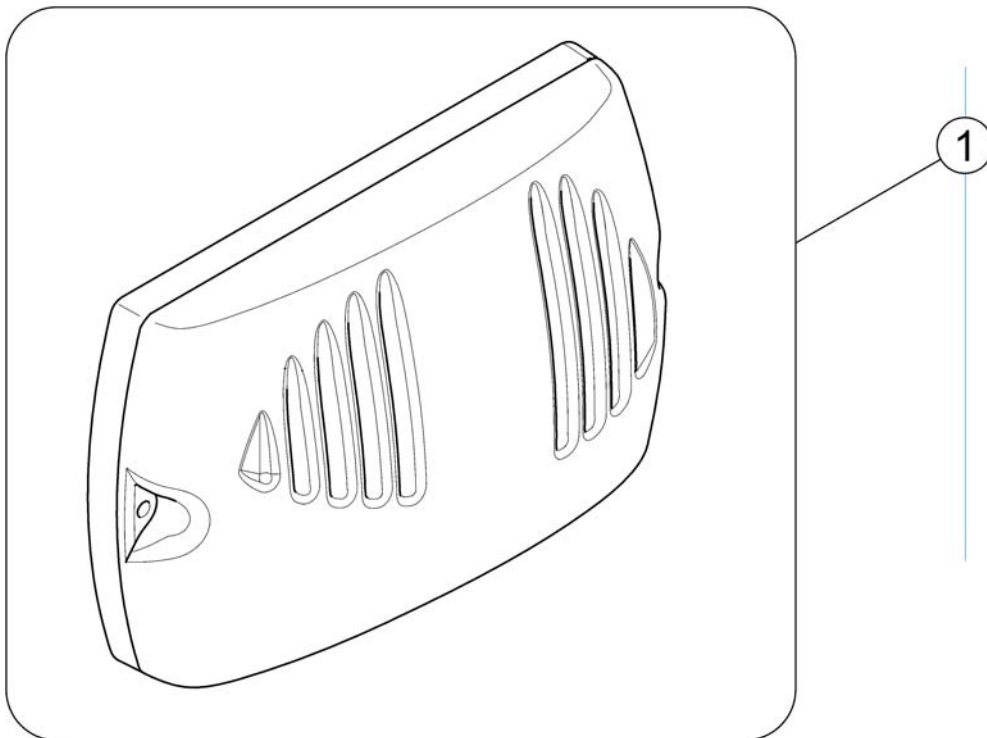

Pos.	Part number	Description	Valid from → until	Qty
1	1136898	Powermodule, Shark, 90A, 2 actuators, DK-PMC28		1

# Bracket for Power Modul


1583357.iso

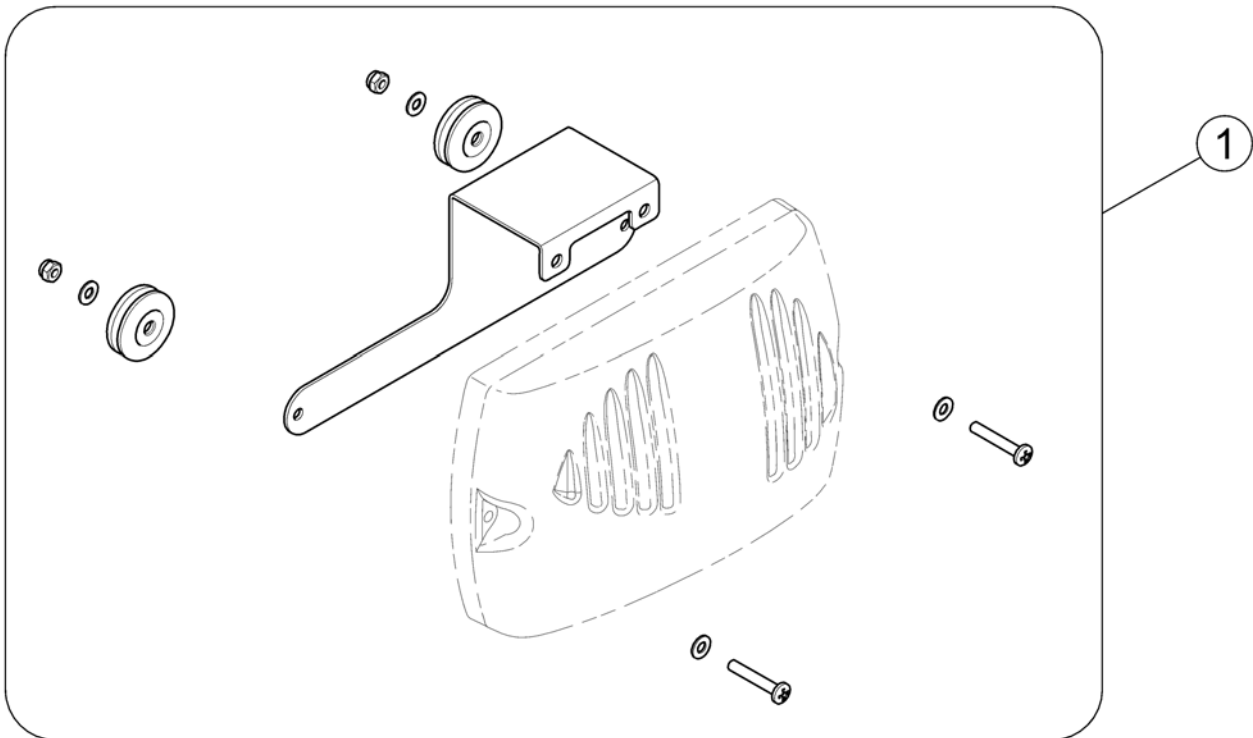

Pos.	Part number	Description	Valid from → until	Qty
1	SP1583358	Bracket for Shark Power Modul		1

## Shark Remote & Brackets


156025 iso

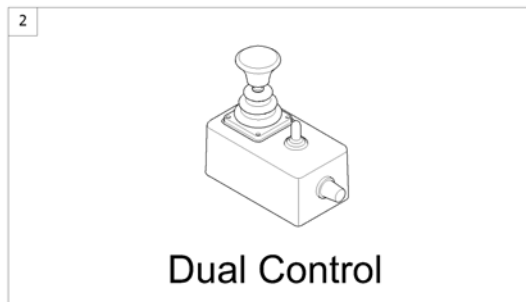

Pos.	Part number	Description	Valid from → until	Qty
1	-	Remote Shark with Brackets <i>[1801]</i>		1
2	-	Remote Dual Control with Brackets <i>[1891/1892]</i>		1

## Remote Shark with Brackets



1563326.iss

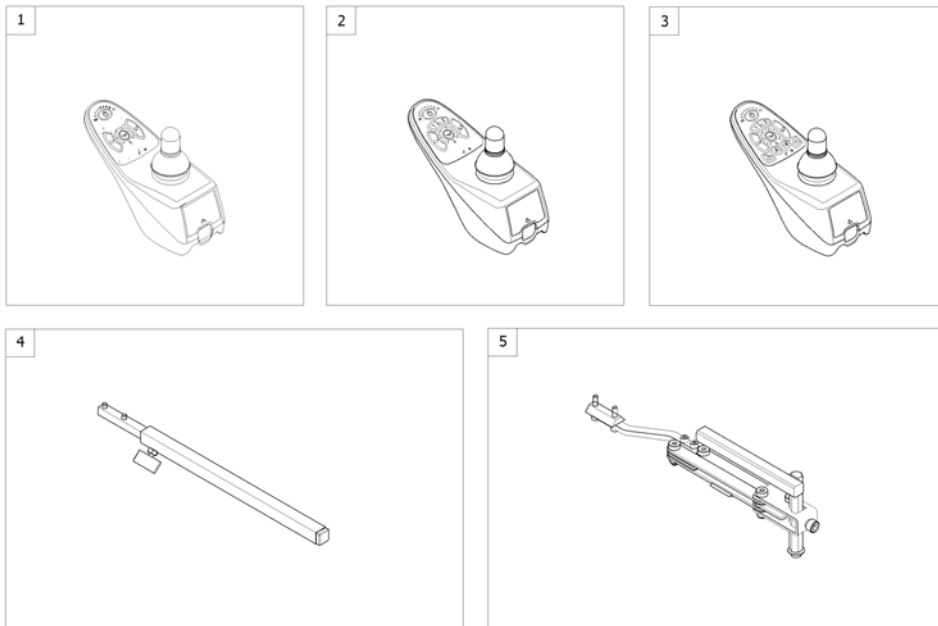


Pos.	Part number	Description	Valid from → until	Qty
1	1492567	Remote DK-REM D01, Shark2 (Drive only) [1801]		1
2	1449177	Remote DK-REM D11, Shark2 (Drive & Adjustment) [1801]		1
3	1484904	Remote DK-REM D21, Shark2 (Drive, Adjustment, Light) [1801]		1
4	-	Remote Bracket Standard [1801+1207/1208]		1
5	-	Remote Bracket Swing away & height adjust. [1801+1707/1708]		1

# Remote Dual Control with Brackets

1563329.iss

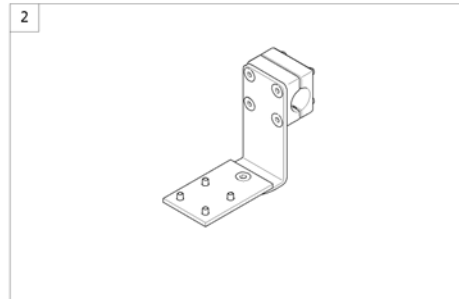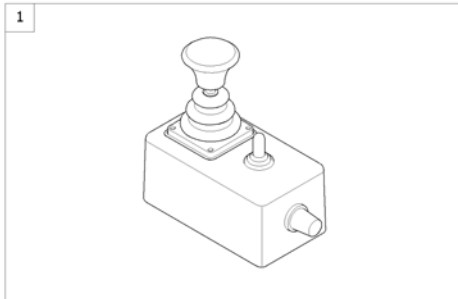

Pos.	Part number	Description	Valid from → until	Qty
1	1444717	Shark Dual Control, switchable <i>[1891/1892]</i>		1
2	-	Remote Bracket Dual Control <i>[1891/1892]</i>		1

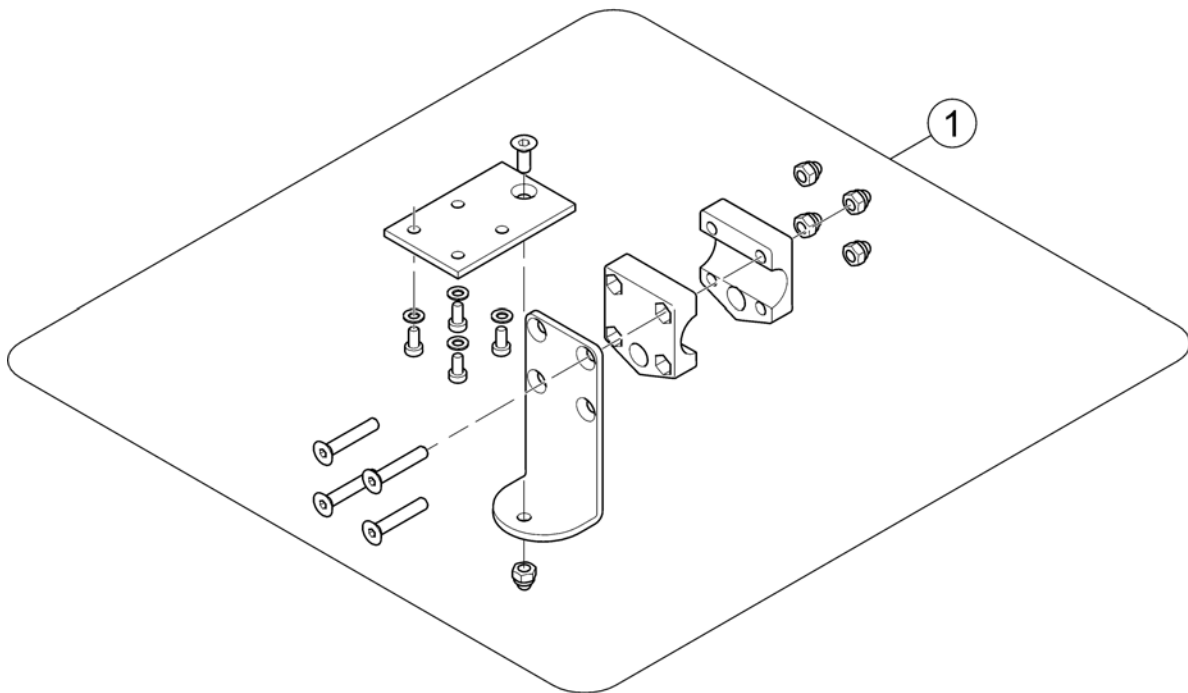

Pos.	Part number	Description	Valid from → until	Qty
1	1450307	Remote Bracket Attendent Control Shark		1


## Shark Cables & Accessories

Pos.	Part number	Description	Valid from → until	Qty
-		Bus-Cables Shark		0
-		Cables Shark		0
-		Programming Kits		0



# Bus-Cables Shark


SHARKBUSKABEL.00

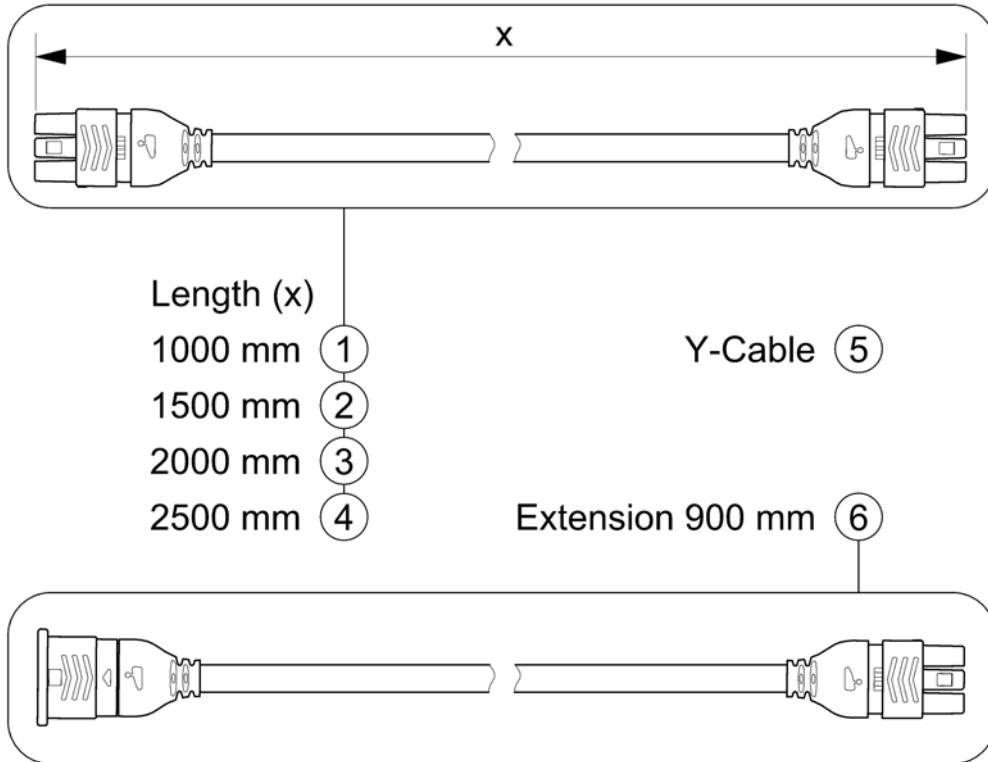

Pos.	Part number	Description	Valid from → until	Qty
1	1448922	Buscable Shark 1000mm		1
2	1450313	Buscable Shark 1500mm		1
3	1484911	Buscable Shark 2000mm		1
4	SP1510073	Buscable Shark 2500mm		1
5	1450314	Y-Cable		1
6	1116404	Extension Cable Shark 900mm		1

# Cables Shark



1583705.000

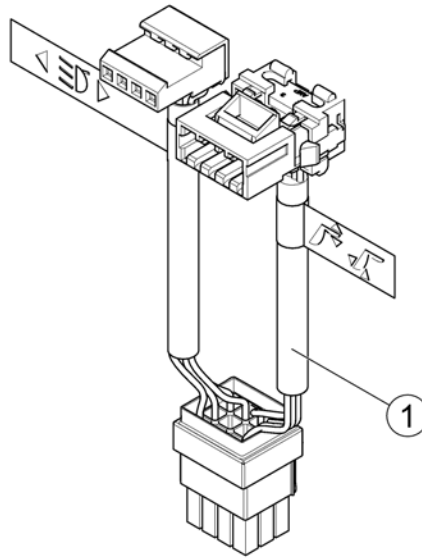

Pos.	Part number	Description	Valid from → until	Qty
1	1488951	Wire Harness Shark - electrical adjustments & light		1


## Batteries, Cables & Chargers

SPCTDX\_SP280.iso

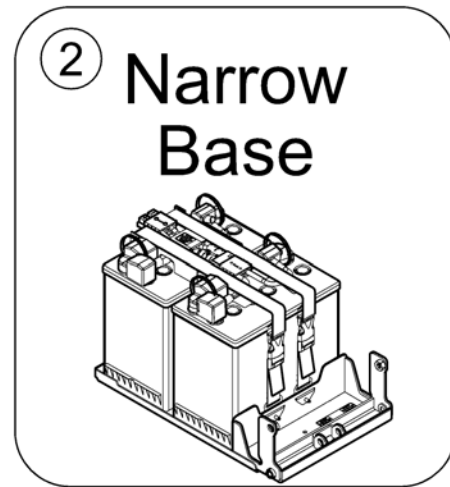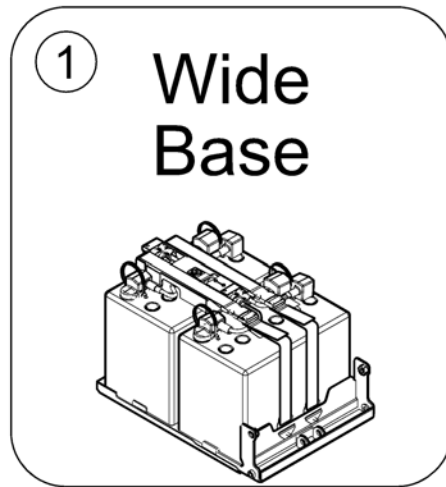

Pos.	Part number	Description	Valid from → until	Qty
1	-	Battery, Tray & Wiring - Wide Base		0
2	-	Battery, Tray & Wire Harness - Narrow Base		0
3	-	Charger		0

## Battery, Tray & Wiring - Wide Base

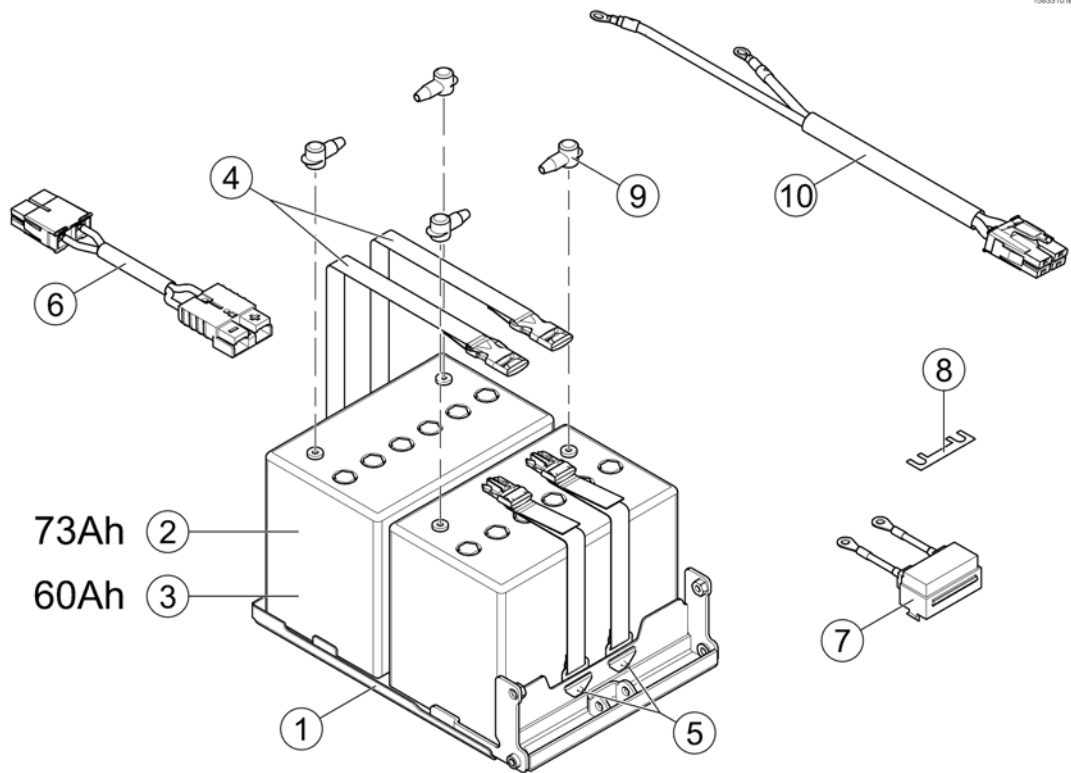


1563310 (no)

Pos.	Part number	Description	Valid from → until	Qty
-	SP1523150	Strip, Self Adhesive Wear (2-1/2"Long x1/2"Wide)		1
1	SP1179723	Kit, Battery Tray Assy - Wide Base		1
2	P50016	Battery, 12V/73,6 Ah C20, Gel		2
3	1432944	Battery, 12V/60Ah C20, GEL <i>Lowrider Option only!</i>		2
4	SP1179296	Battery Strap Assy EU, GP24		2
5	SP1523266	Rivet (3/16 x7/16")		4
6	SP1527780	Battery Cable ACS2		1
7	1432561	Strip Fuse Holder with 80A Fuse		1
8	1433734	Strip Fuse 80A		1
9	P52996	Battery terminal cover		4
10	-	Not available so far.		0

## Battery, Tray & Wire Harness - Narrow Base


1583311.00

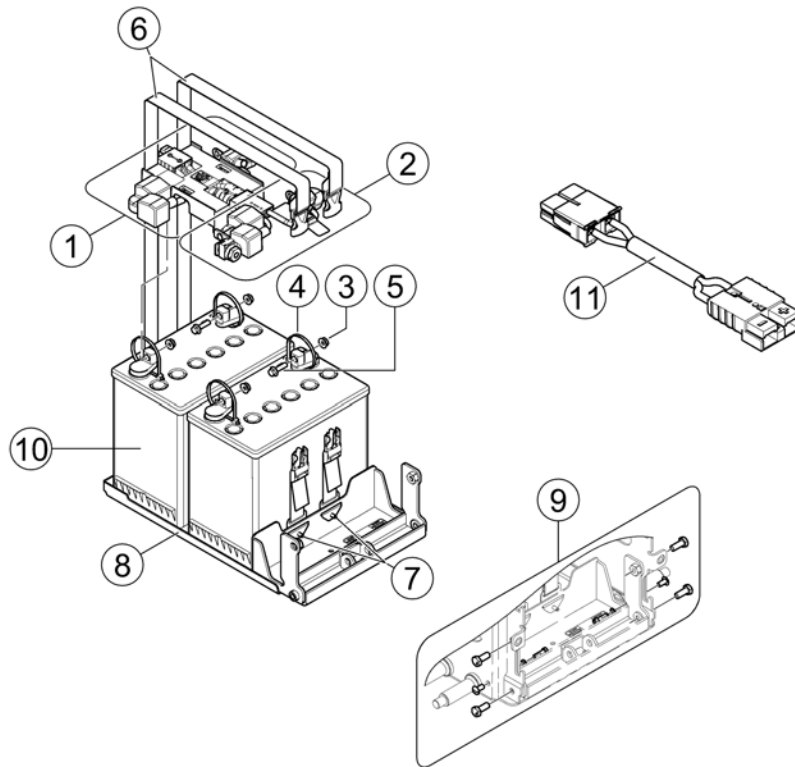

Pos.	Part number	Description	Valid from → until	Qty
-	SP1523150	Strip, Self Adhesive Wear (2-1/2"Long x1/2"Wide)		1
1	SP1120363	Kit, Battery Wiring w/Terminal Hardware, rear		1
2	SP1120362	Kit, Battery Wiring w/Terminal Hardware, front		1
4	SP1522970	Package, Cable Ties (11-1/2"Long)		1
5	SP1522972	Screw, Hex Flange Whiz Lock (1/4-20x7/8")		4
6	SP1144682	Battery strap		1
7	SP1523266	Rivet (3/16 x7/16")		4
8	SP1179722	Kit, Battery Tray Assy - Narrow Base		1
9	SP1179896	Kit, Battery Tray Stop Hardware		1
10	PG50989	Battery, 12V/50,9Ah C20, GEL		2
11	SP1527780	Battery Cable ACS2		1

# Charger



1583312.00

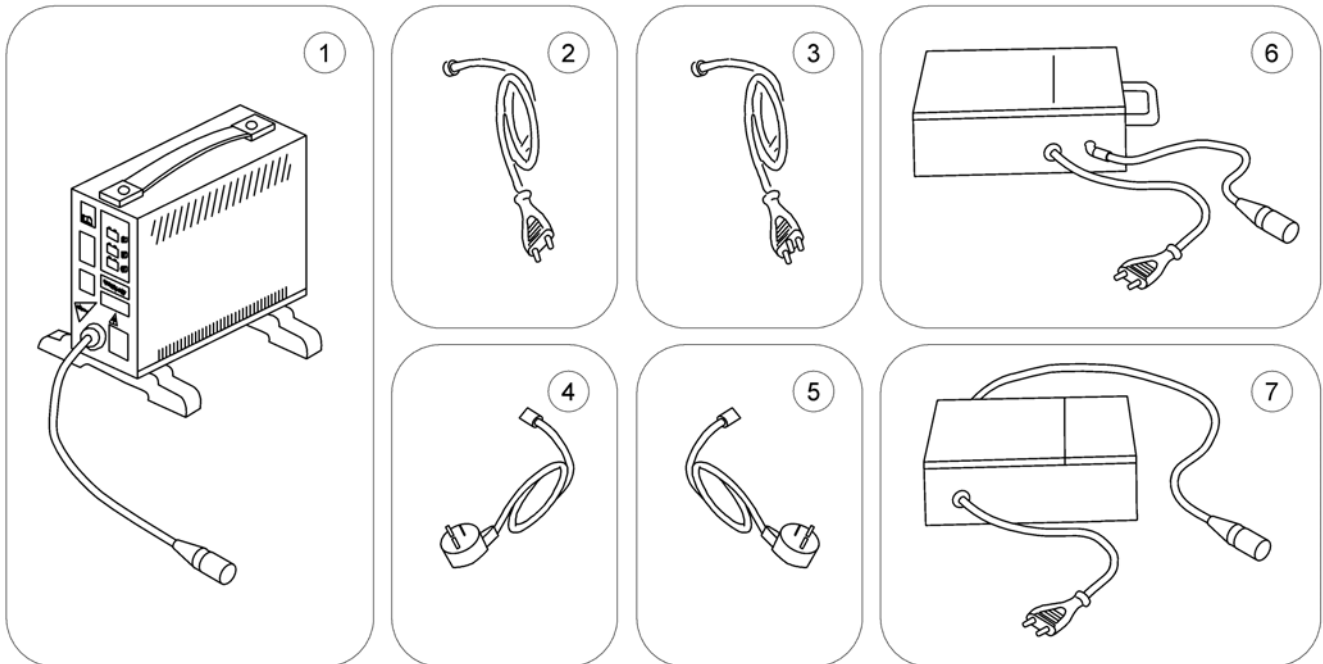

Pos.	Part number	Description	Valid from → until	Qty
1	SPP51981	Charger 8A for Batteries up to 73Ah (without power cord) [1343]		1
2	P21079	Power Cord, Continental Shockproof Plug [for 1343]		1
3	1443841	Power Cord, Swiss [for 1343]		1
4	P21080	Power Cord, UK [for 1343]		1
5	SP1553849	Power Cord, Israel [for 1343]		1
6	5813987	Charger 8A, Gi Popular (Sweden only) [1388]		1
7	SP1545511	Charger 10A, LADAC, incl. power cord continental shockproof [1389]		1

# Lightsystem


1582964.iso

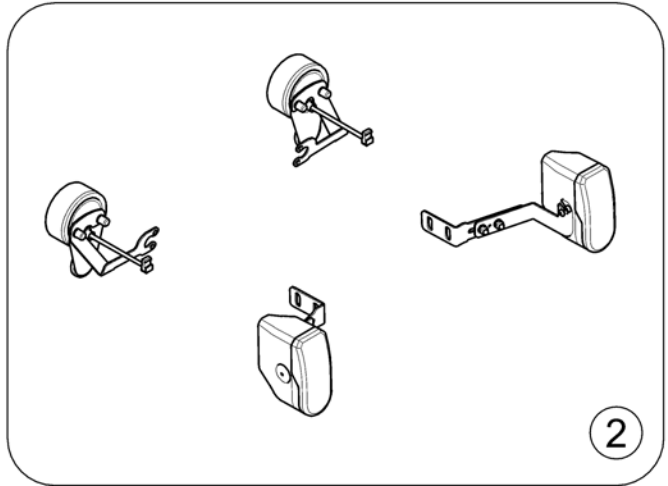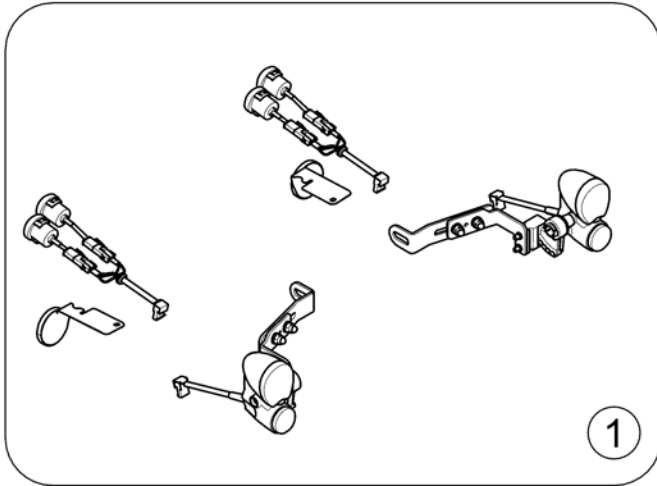

Pos.	Part number	Description	Valid from → until	Qty
1	-	Lightsystem LED [1437]		1
2	-	Lightsystem german Homologation [1367]		1

# Lightsystem LED

1583076 iso

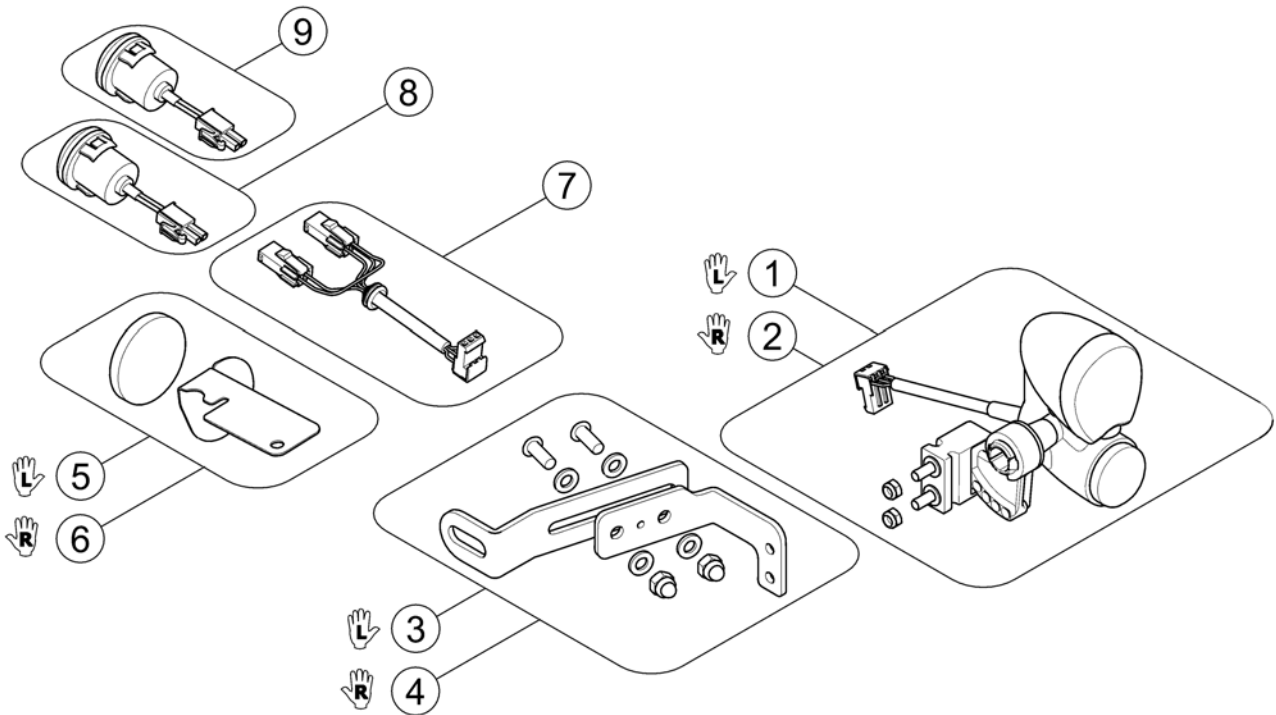


Pos.	Part number	Description	Valid from → until	Qty
1	SP1536030	Front light LED, LH		1
2	SP1576193	Front Light LED RH		1
3	SP1583093	Light Bracket LED, front LH		1
4	SP1583092	Light Bracket LED, front RH		1
5	SP1583095	Reflector rear LH		1
6	SP1583094	Reflector rear RH		1
7	SP1529712	Rear light LED-Cable		1
8	SP1530288	Rear light		1
9	SP1530287	Turn indicator		1



## Lightsystem german Homologation


1583077.00

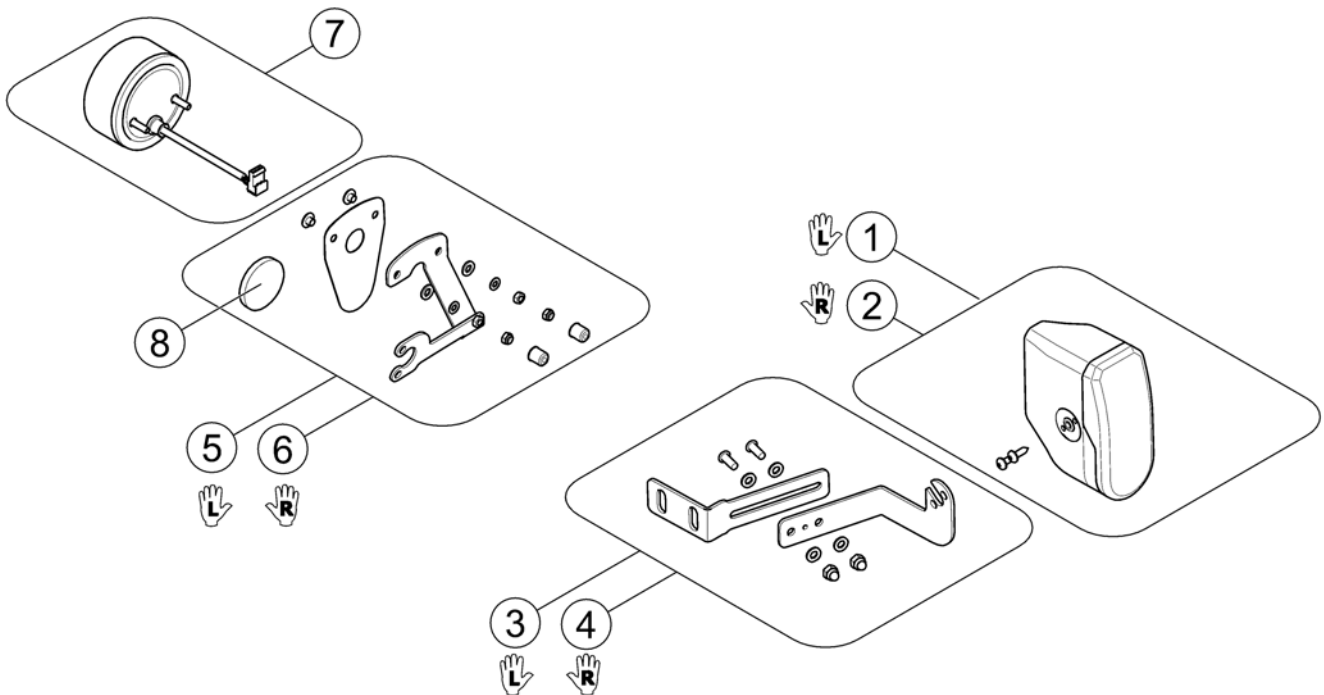

Pos.	Part number	Description	Valid from → until	Qty
-	P31108	Light Bulb 24V/10W (BA15s), white (Front)		1
-	P31191	Light Bulb 24V/5W, white (Rear)		1
-	SP1419271	Light Bulb 24V/21W, yellow (Turn Signal front/rear)		1
!!!	-	Rear Light Protector Kit		0
1	1440426	Headlight 24V, LH, cable 800mm, without bracket		1
2	1420412	Headlight 24V, RH, cable 900mm, without bracket		1
3	SP1583088	Light Bracket front LH		1
4	SP1583087	Light Bracket front RH		1
5	SP1583089	Light Bracket rear LH		1
6	SP1583090	Light Bracket rear RH		1
7	1442736	Rear Light 24V, cpl.		1
8	SP1541591	Reflector, red		1

# Rear Light Protector Kit


1585420.000

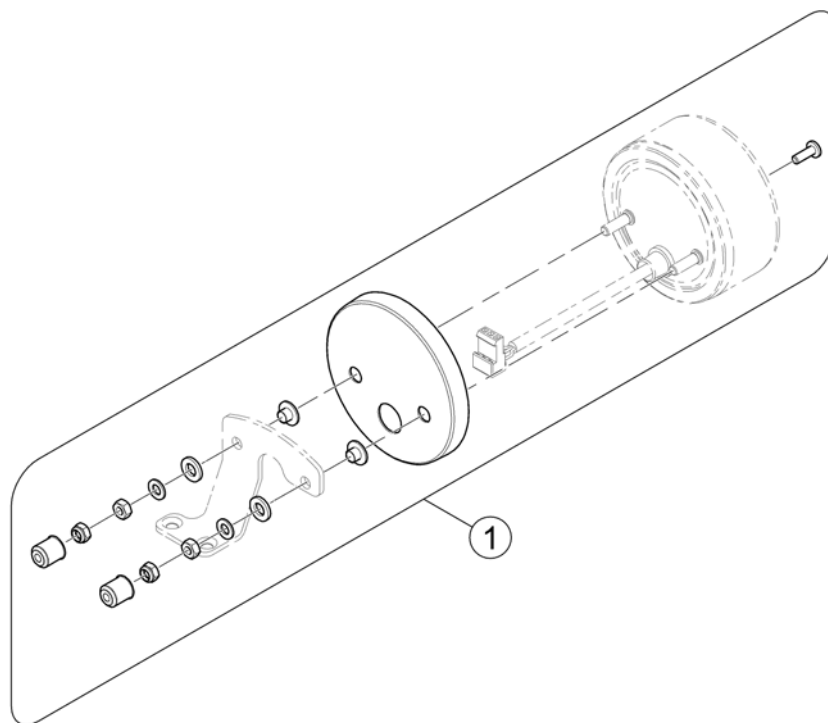

Pos.	Part number	Description	Valid from → until	Qty
1	SP1581827	Rear Light Protector Exchange Kit		1

# Reflective Sign


SP1430410.iss

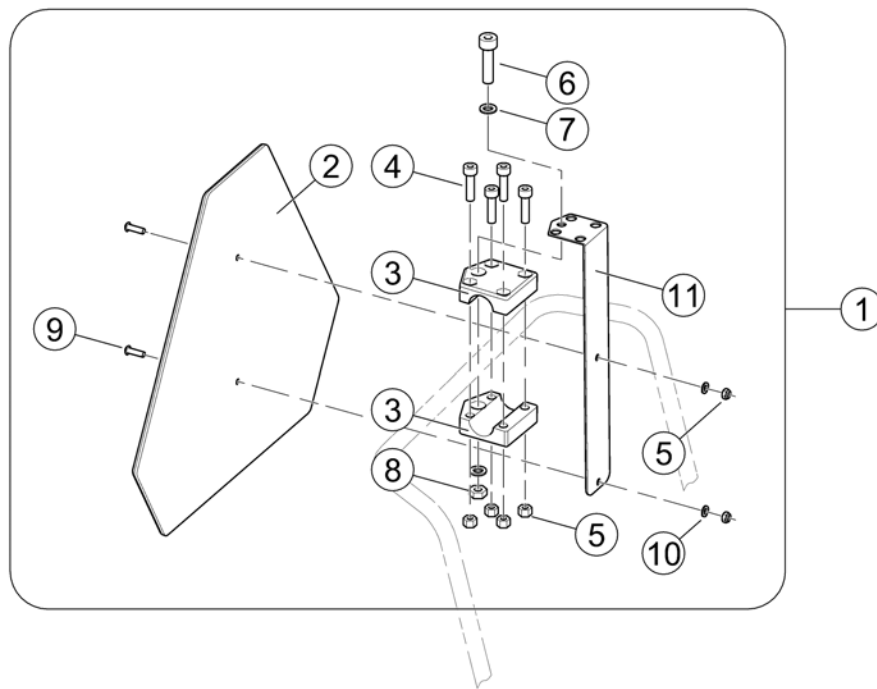

Pos.	Part number	Description	Valid from → until	Qty
1	SP1430410	Reflective Sign ECE-R 69, complete		1
2	1430834	Rear Marking Sign Ece-R 69		1
3	Z12008A.P40	Gripping clip with hexagon hole, RAL 9005 black		2
4	D50004	Cylinder Bolt M6x30-8.8 bk - DIN 912		4
5	F20109	Hexagon Nut M6 bk - DIN 985		6
6	D00102	Hexagon Bolt M8x45-8.8 bk - DIN 931		1
7	G00107	Washer Ø8,4 bk - DIN 125		2
8	F00002	Hexagon Nut M8 bk - DIN 985		1
9	D90187	Fillister Head Screw M6x20-10.9 bk - DIN 7380		2
10	G00115	Washer Ø6,4 bk - DIN 125		2
11	1430418-9005	Mounting Bracket for Reflective Sign - RAL 9005 black		1
12	SP1580671	Bracket, Reflective Sign, cpl.		1

## Adapter-Kit DAHL Docking Station - Wide Base


1583706 iso

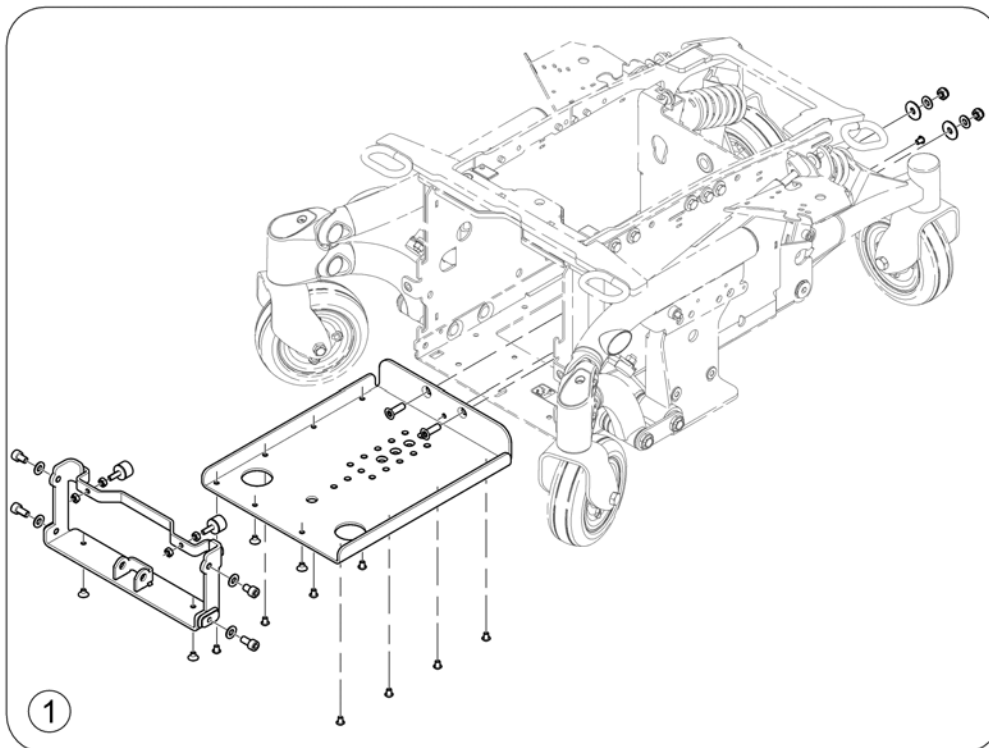

Pos.	Part number	Description	Valid from → until	Qty
1	SP1564843	Retrofit-Kit for DAHL Docking Station - Wide Base		1

## Adapter-Kit DAHL Docking Station - Narrow Base

1583707 iso

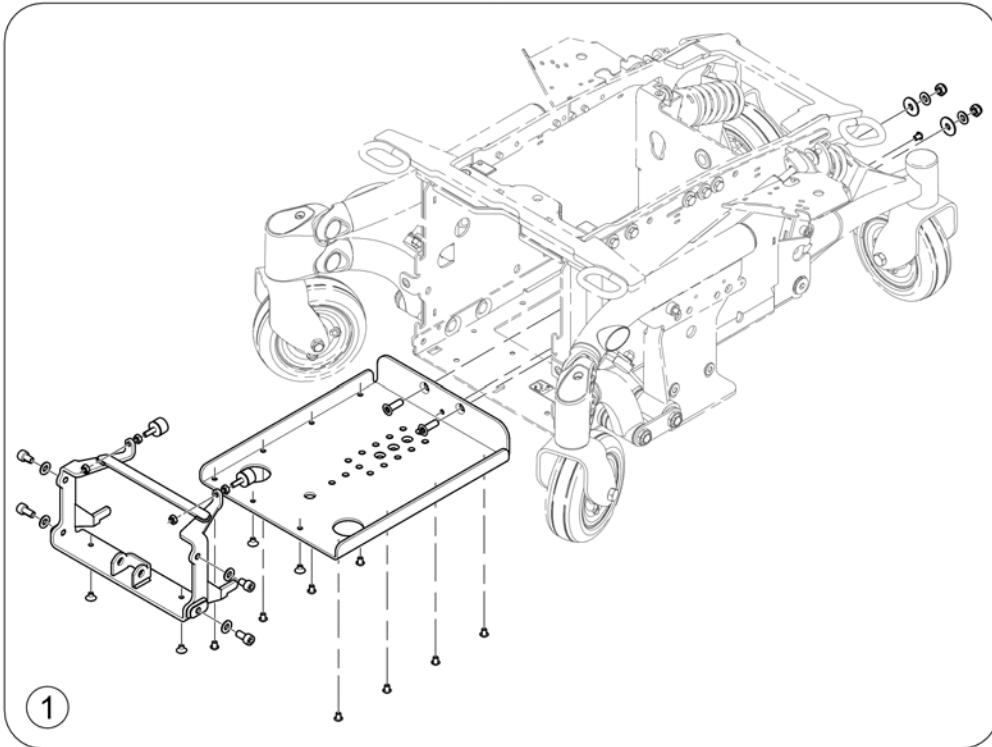


Pos.	Part number	Description	Valid from → until	Qty
1	SP1564807	Retrofit-Kit for DAHL Docking Station - Narrow Base		1

## Seat belts

Pos.	Part number	Description	Valid from → until	Qty
-		Lap Belts (Modulite) <i>[1329/2113]</i>		1
-		Lap Belt (Max Seat)		1
-		H-Belts (Modulite) <i>[1332]</i>		1

## Lap Belts (Modulite)


1571836 iso

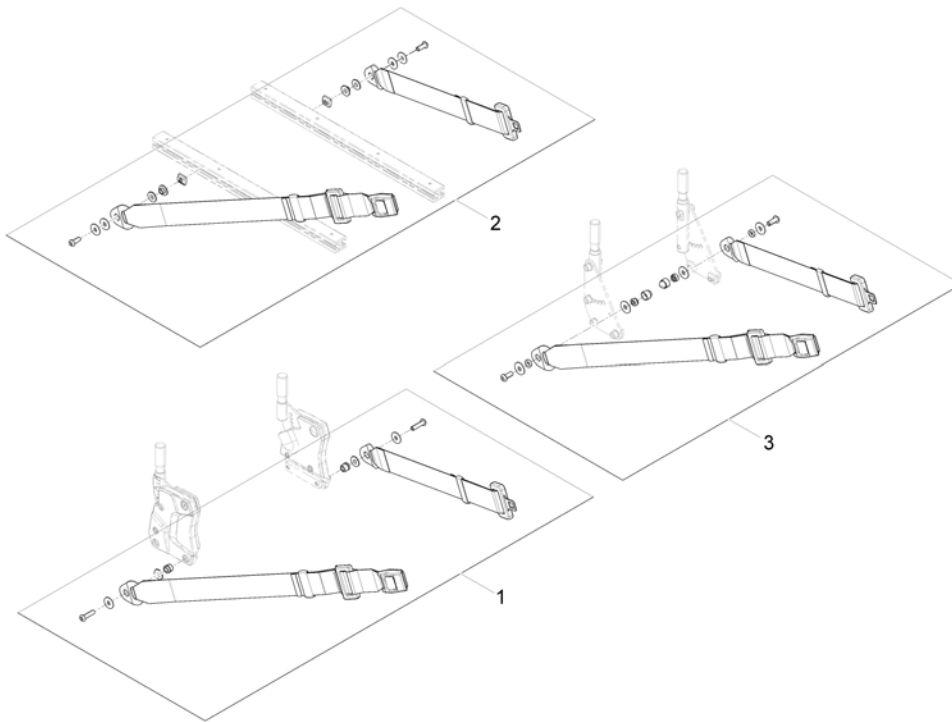

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570381	Lap Belt (fix mounting on Backrest Support), cpl.		1
2	SP1571183	Lap Belt (flexible mounting in side rails), cpl.		1
3	1422845	Lap Belt, cpl.		1

## Lap Belt (Max Seat)


1582371.00

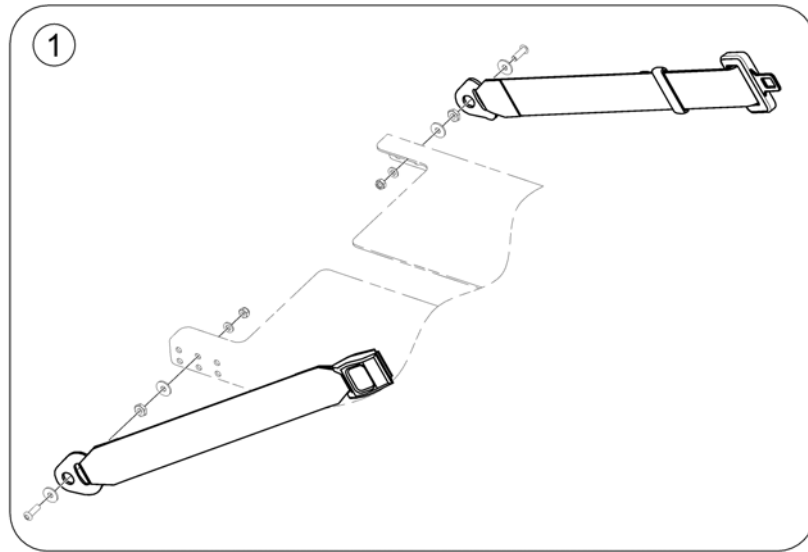

Pos.	Part number	Description	Valid from → until	Qty
1	1551067	Lap Belt Autolock, long		1




## H-Belts (Modulite)

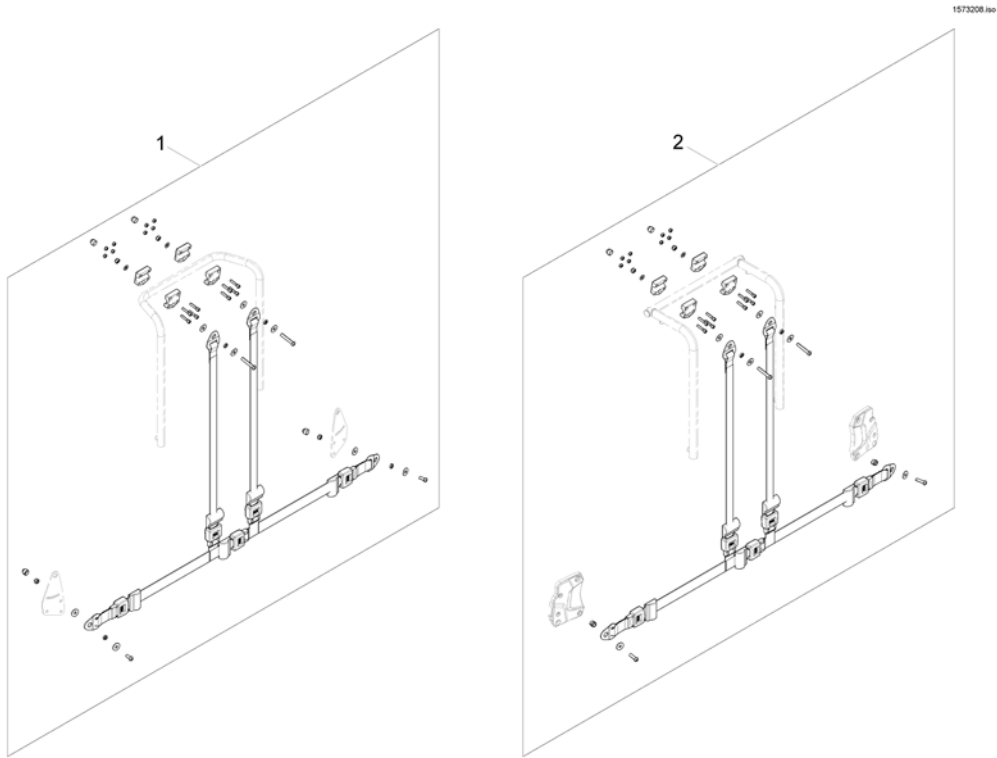


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570385	H-Belt for Backrest Frame		1
2	SP1570380	H-Belt for Back Posts		1

## Rearview Mirror

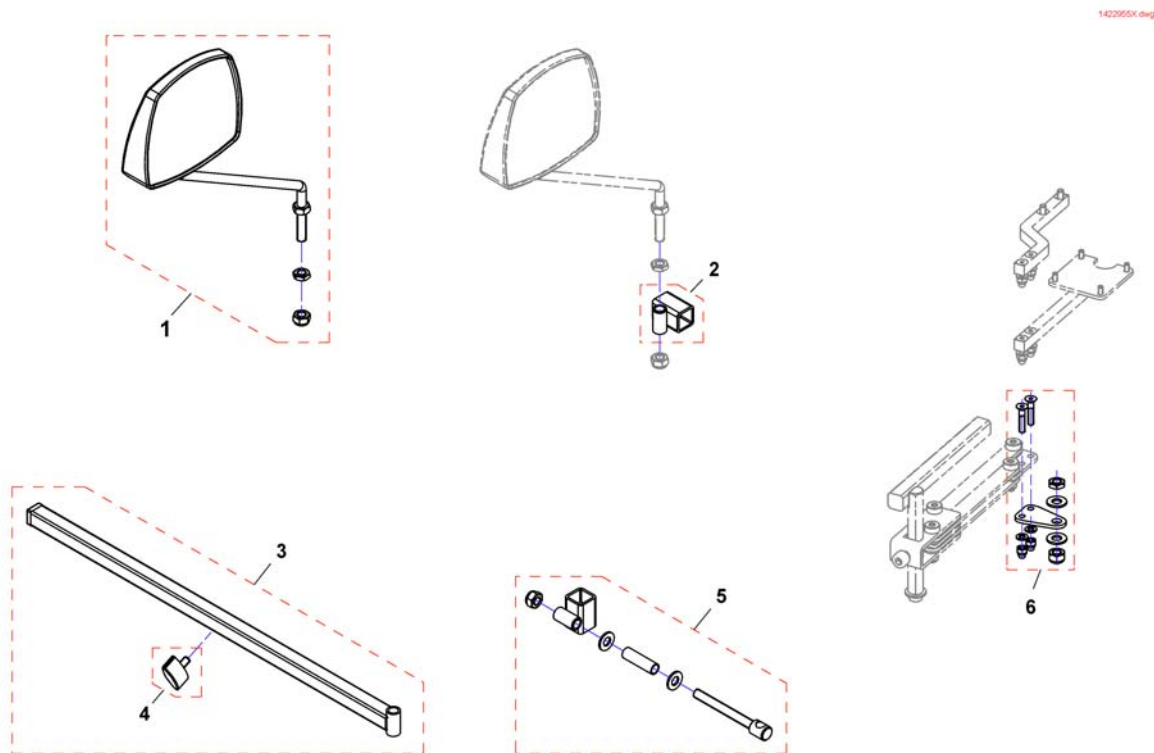


Pos.	Part number	Description	Valid from → until	Qty
1	SP1422955	Rear Mirror		1
2	F276005.00	Mirror bracket		1
3	SP1423435	Rear Mirror Support Rail cpl.		1
4	L31001	Wing ScREW M6X10		1
5	SP1422957	Mirror bracket		1
6	SP1527151	Adapter to mount mirror and remote at the same side		1

## Head- / Neckrest

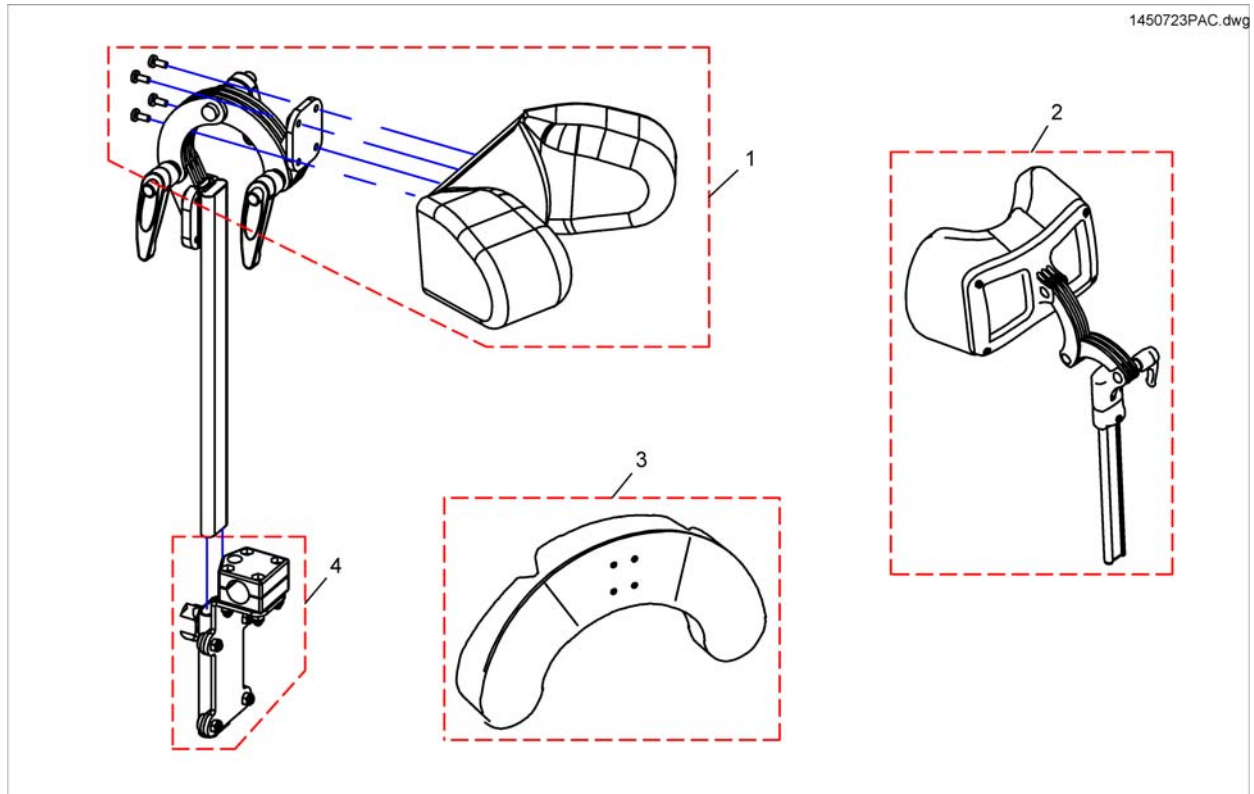

Pos.	Part number	Description	Valid from → until	Qty
1	1487404	Headrest cushion VCR, cpl.		1
2	SP1527050	Neckrest, depth adjustable		1
3	1444304	Cushion, cheek support		1
4	SP1422914	Head-/Neckrest Bracket, cpl.		1


## Headrest sidewise adjustable

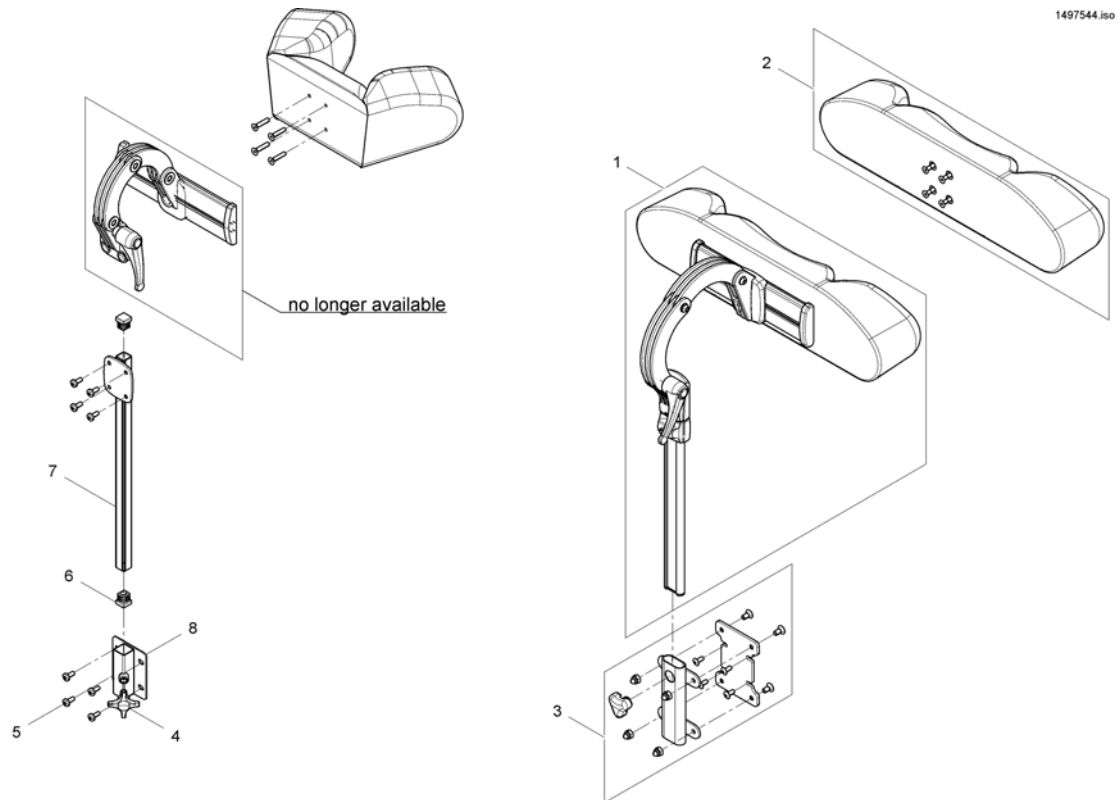

Pos.	Part number	Description	Valid from → until	Qty
1	SP1530119	Headrest, sidewise adjustable, cpl. - Dartex TR26		1
2	SP1422865	Headrest cushion contour black, without bracket		1
2	1422866	HEADREST PAD STD./CONTOUR GREY		1
2	SP1522214	HEADREST PAD STD./CONTOUR BLACK PLUSH		1
3	SP1544680	MOUNTING-KIT		1
4	256099	Wing Screw M8 X 20 639		1
5	D30197	Lens head screw DIN7985-M05x12-4.8 zc		1
6	L01055	Plug 15x15mm		1
7	1451639-0205	Headrest REA Angle Adjustment		1
8	26002351	Tube Headrest		1

## Headrest with cheek support


1446560X.dwg

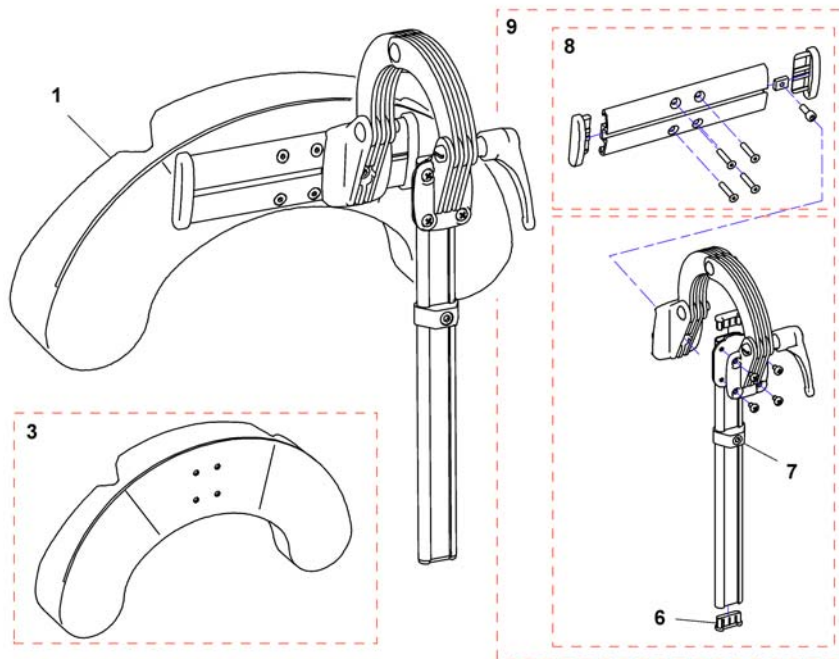

Pos.	Part number	Description	Valid from → until	Qty
1	1446560	HEAD/CHEEK SUPP. TR26 CPL. SIDEWISE ADJUSTABLE		1
1	SP1541658	Head/cheek support TR26 cpl. sidewise adjustable EasyAdapt		1
3	1444304	Cushion, cheek support		1
5	SP1529572	Headrest holder for EasyAdapt		1
6	5716497	Plug IPO "special"		1
7	1448661	stop neck and headrest		1
8	1445838	Rail head support		1
9	1485538	Headrest fixation kit		1

# Trunk Support - Modulite


1572155.iso

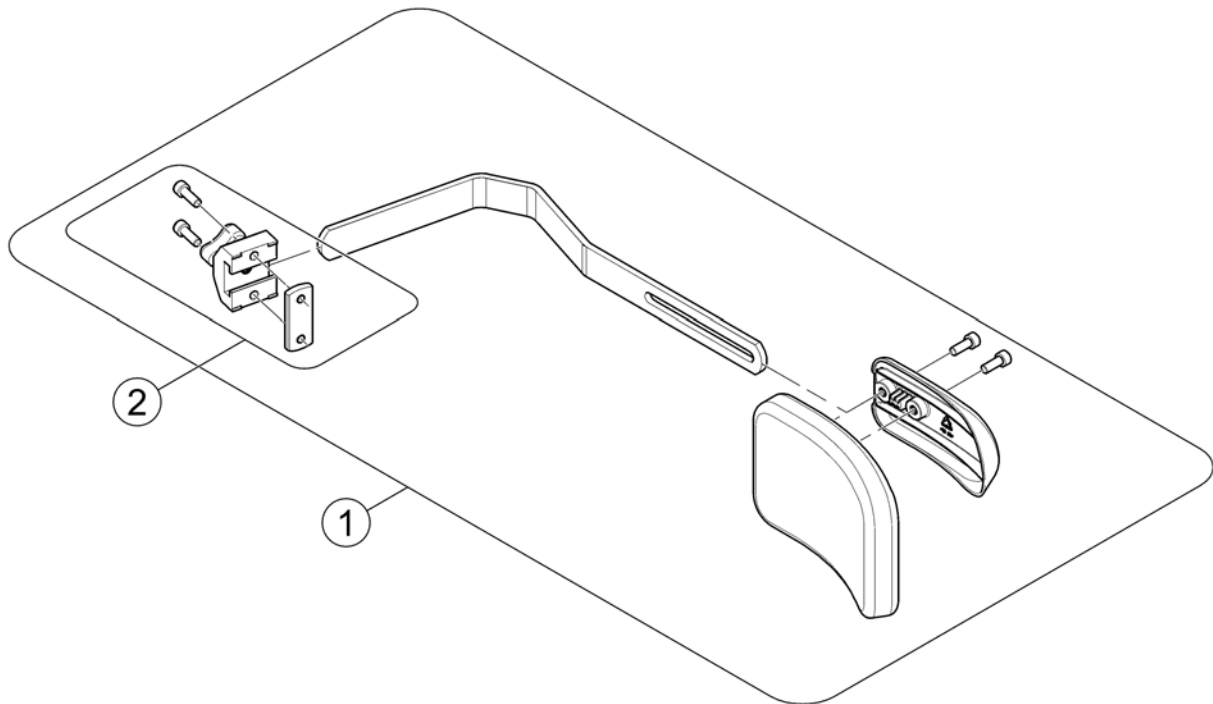

Pos.	Part number	Description	Valid from → until	Qty
1	SP1580394	Trunk Support Modulite, cpl Cushion		1
2	1543574	Locking trunk support		1

# Trunk Support, with extended Arm - Modulite

1562647.iss

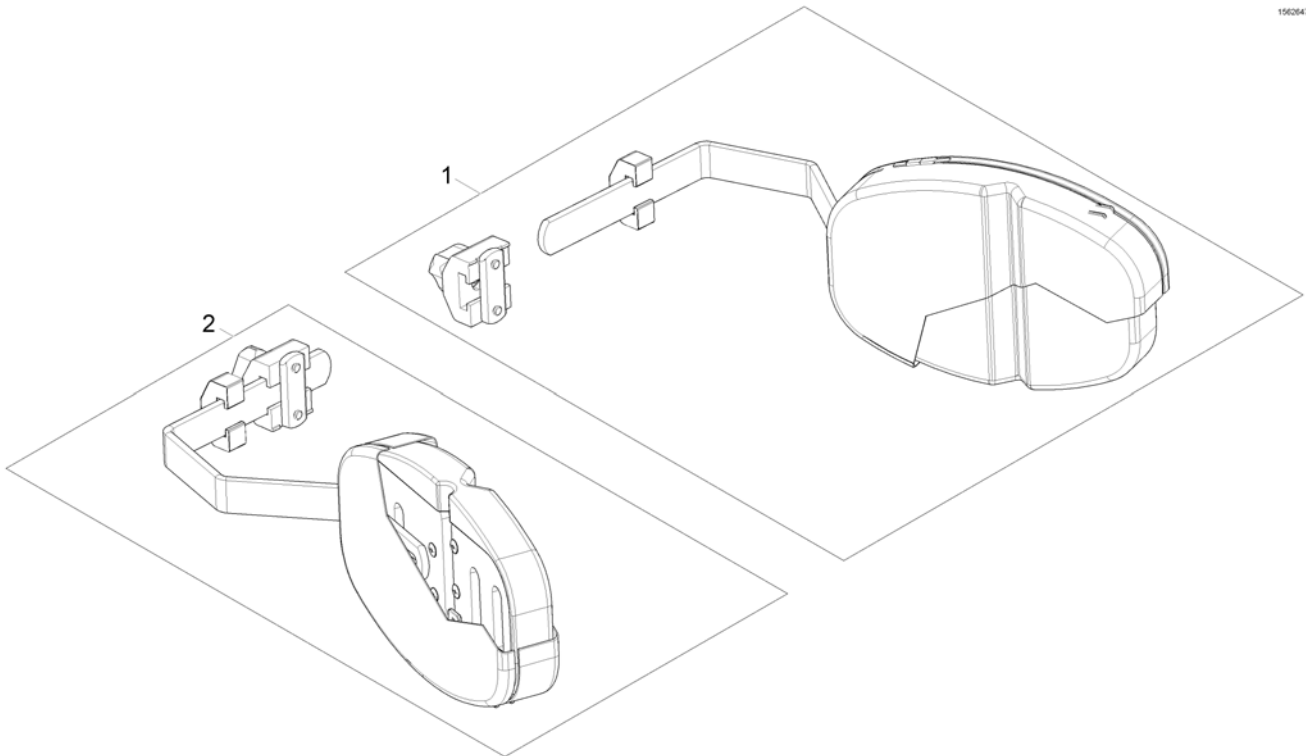


Pos.	Part number	Description	Valid from → until	Qty
1	1448692	Trunk Support, Multi Adjustable, LH, Dartex Grey, cpl. with Holder		1
2	1448693	Trunk Support, Multi Adjustable, RH, Dartex Grey, cpl. with Holder		1

## Cane holder

Pos.	Part number	Description	Valid from → until	Qty
-		Cane holder (Modulite: backrest frame)		1
-		Cane holder (Modulite: back posts man recliner)		1
-		Cane holder (Modulite: back posts pow recliner)		1
-		Cane holder (MAX Seat)		1




## Cane holder (Modulite: backrest frame)

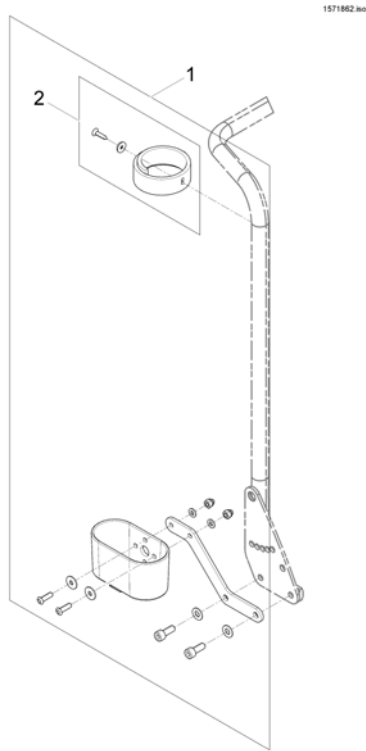


Pos.	Part number	Description	Valid from → until	Qty
1	SP1570396	Cane holder, standard backrest		1
2	SP1420493	Belt for caneholder		1

## Cane holder (Modulite: back posts man recliner)

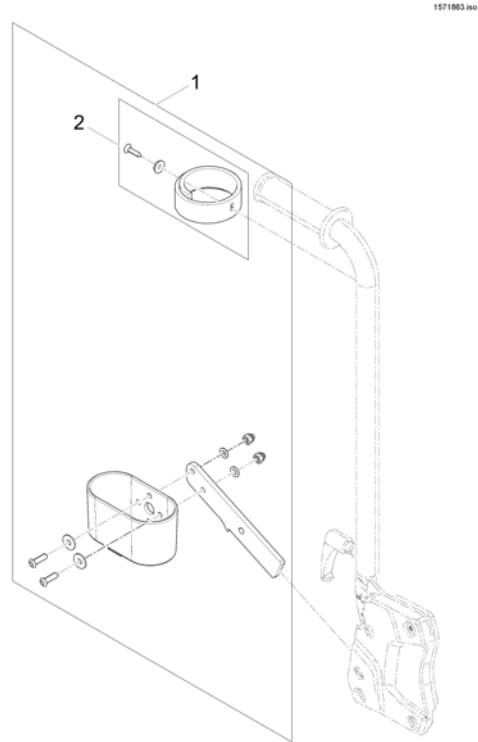

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570443	Cane holder, manual backrest		1
2	SP1420493	Belt for caneholder		1


## Cane holder (Modulite: back posts pow recliner)

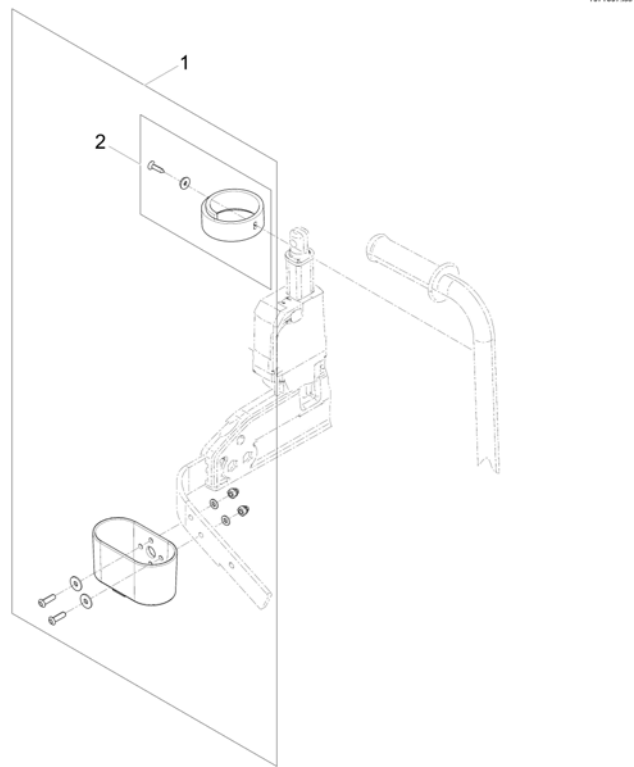

Pos.	Part number	Description	Valid from → until	Qty
1	SP1570382	Cane holder, powered backrest		1
2	SP1420493	Belt for caneholder		1


## Cane holder (MAX Seat)

1567255.iso

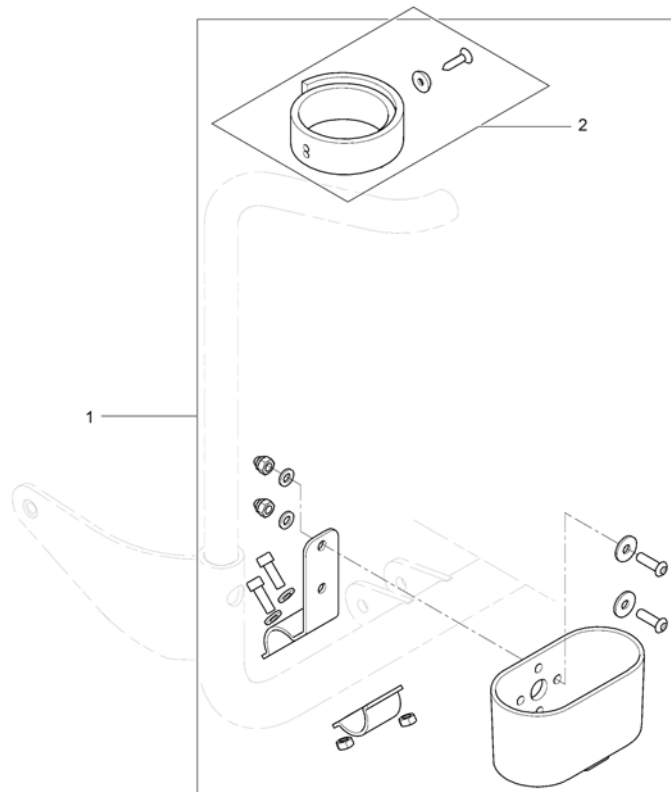

Pos.	Part number	Description	Valid from → until	Qty
1	SP1543078	Caneholder, cpl.		1
2	SP1420493	Belt for caneholder		1

# Trays

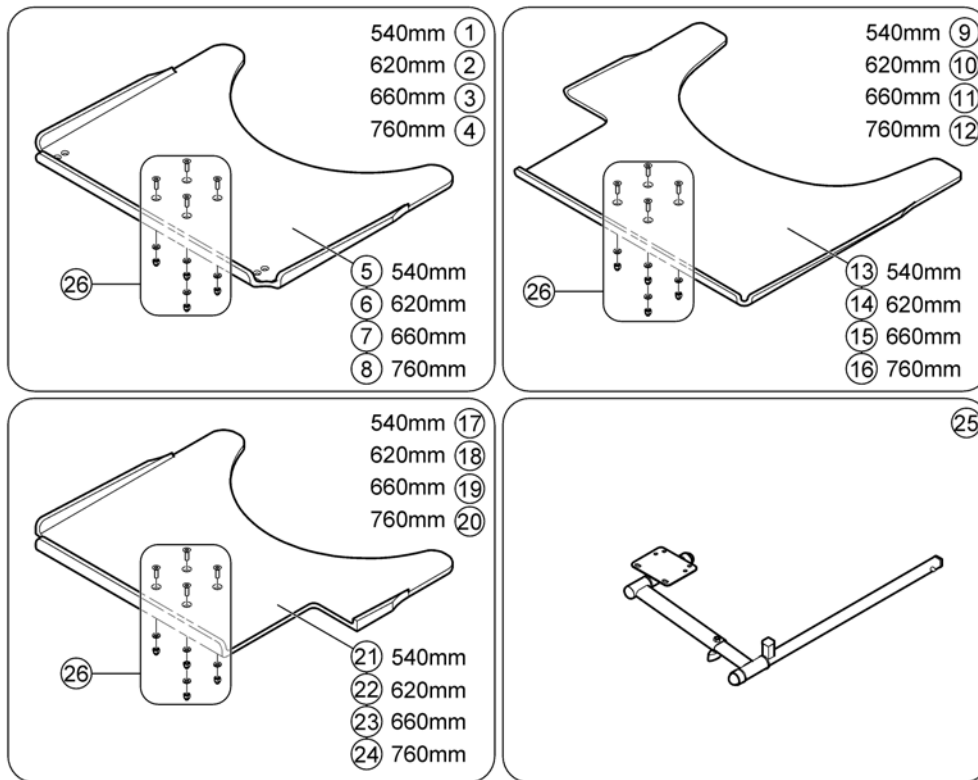

1556046.iso


Pos.	Part number	Description	Valid from → until	Qty
1	SP1548198	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, no Cutout, cpl. with holder		1
2	SP1548201	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, no Cutout, cpl. with holder		1
3	SP1422954	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, no Cutout, cpl. with holder		1
4	SP1584644	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, no Cutout, cpl. with holder		1
5	SP1535050	Tray table, 540mm, no Cutout		1
6	SP1535010	Tray table, 620mm, no Cutout		1
7	SP1535011	Tray table, 660mm, no Cutout		1
8	SP1586049	Tray table, 760mm, no Cutout		1
9	SP1548138	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout RH, cpl. with holder		1
10	SP1548199	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout RH, cpl. with holder		1
11	SP1422953	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout RH, cpl. with holder		1
12	SP1584663	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout RH, cpl. with holder		1
13	SP1535051	Tray table, 540mm, Cutout RH		1
14	SP1535012	Tray table, 620mm, Cutout RH		1
15	SP1535049	Tray table, 660mm, Cutout RH		1
16	SP1586048	Tray table, 760mm, Cutout RH		1
17	SP1548197	Swing-Out Tray, 540mm, Seatwidth 330mm to 430mm, Cutout LH, cpl. with holder		1
18	SP1548200	Swing-Out Tray, 620mm, Seatwidth 380mm to 480mm, Cutout LH, cpl. with holder		1
19	SP1422952	Swing-Out Tray, 660mm, Seatwidth 430mm to 530mm, Cutout LH, cpl. with holder		1
20	SP1584662	Swing-Out Tray, 760mm, Seatwidth 480mm to 580mm, Cutout LH, cpl. with holder		1
21	SP1535009	Tray table, 540mm, Cutout LH		1
22	SP1534974	Tray table, 620mm, Cutout LH		1

## Trays

Pos.	Part number	Description	Valid from → until	Qty
23	SP1534975	Tray table, 660mm, Cutout LH		1
24	SP1586047	Tray table, 760mm, Cutout LH		1
25	SPZ177011	Tray Support Fixing Kit cpl.SW 380/505 mm		1
26	Z177025	Fastening-Kit Tray		1

# Hour Counter

1567259.iso

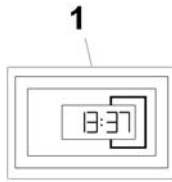

Pos.	Part number	Description	Valid from → until	Qty
1	SP1541531	Hour Counter		1


## KLICKfix® bag & brackets

Pos.	Part number	Description	Valid from → until	Qty
-		KLICKfix® bag		1
-		KLICKfix® bracket for T-Armrest		1
-		KLICKfix® bracket for Fixed Armrest		1
-		KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Module) <i>[1478/1479]</i>		1



# KLICKfix® bag


1562500 iso

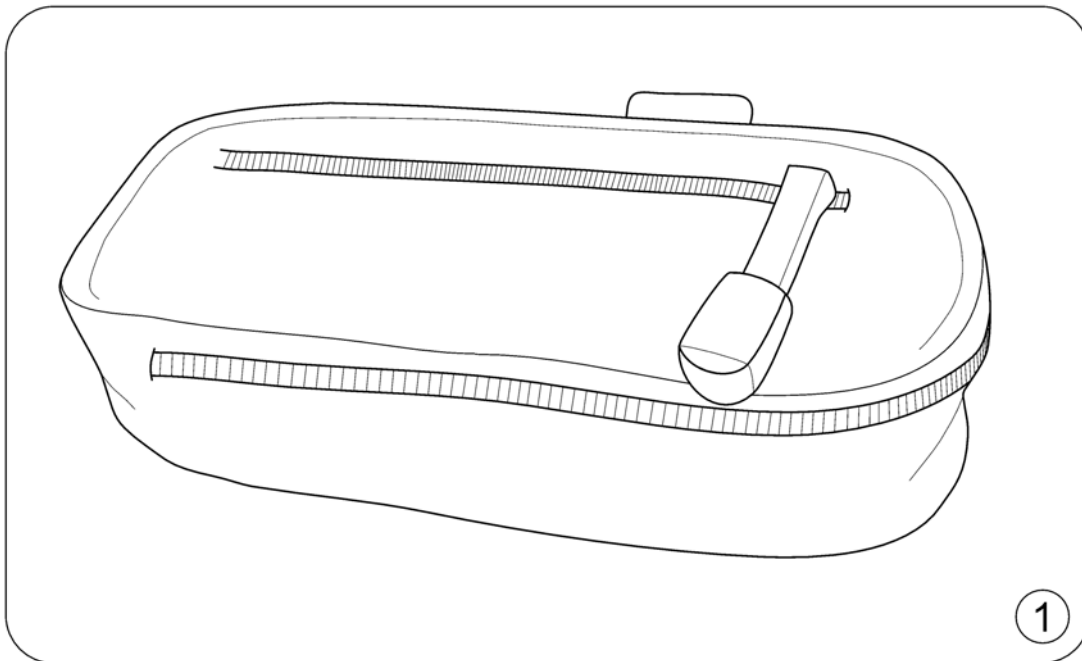

Pos.	Part number	Description	Valid from → until	Qty
1	SP1557532	KLICKfix® bag		1

# KLICKfix® bracket for T-Armrest


1571871 iso

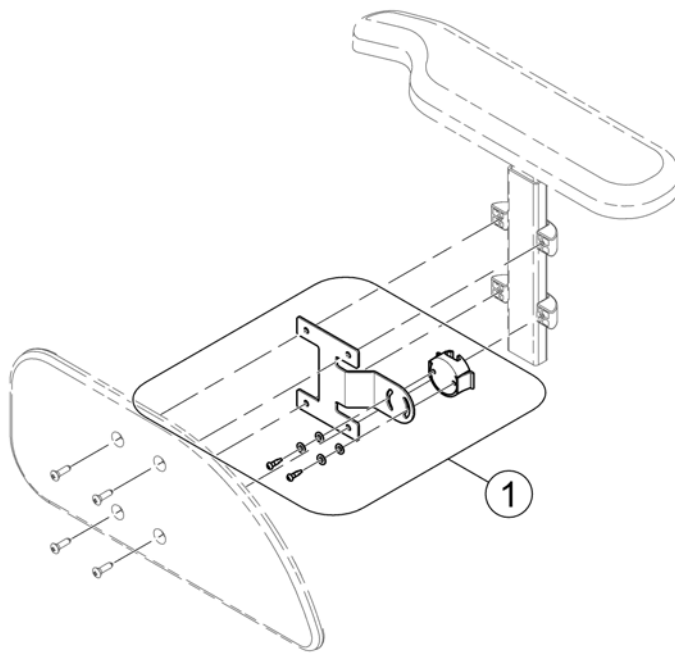

Pos.	Part number	Description	Valid from → until	Qty
1	SP1571678	KLICKfix® bracket		1

# KLICKfix® bracket for Fixed Armrest


1562515.iss

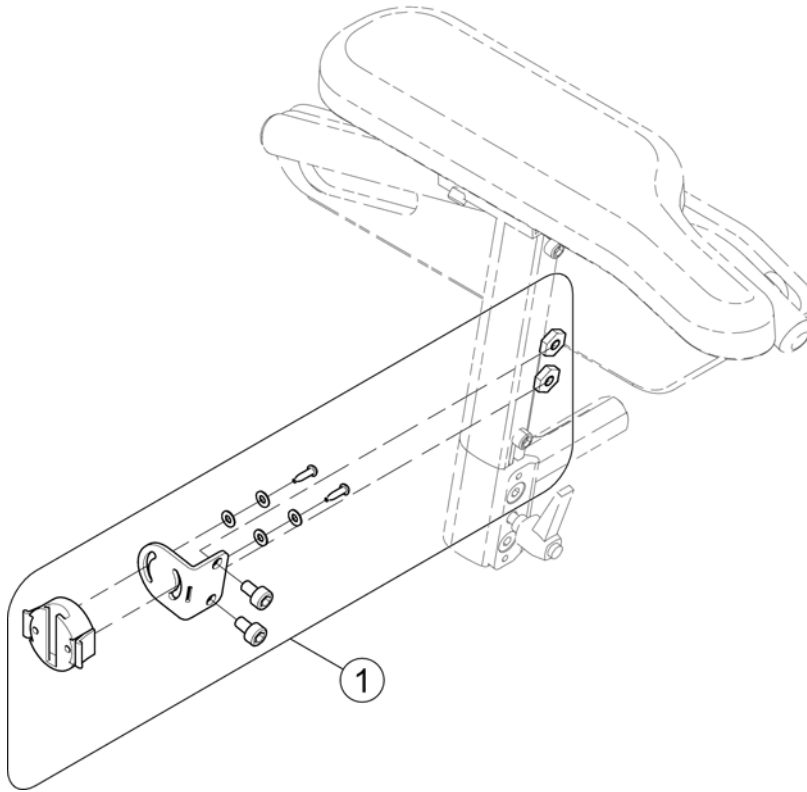

Pos.	Part number	Description	Valid from → until	Qty
1	SP1554320	KLICKfix bracket		1

# KLICKfix® bracket for Inrigged/Flip-Up/Following Armrest (Modulite)


1580167 Rev

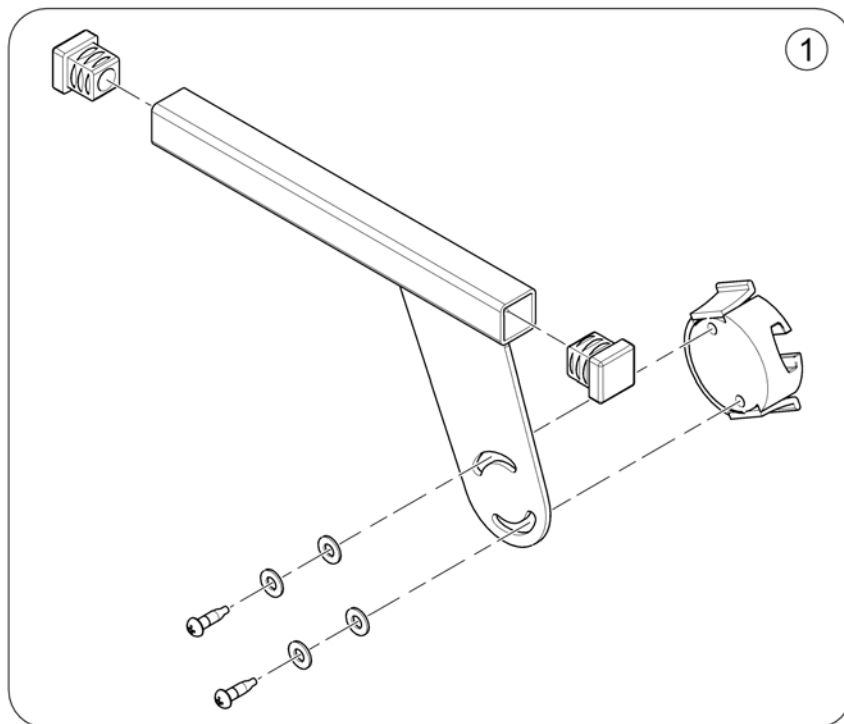

Pos.	Part number	Description	Valid from → until	Qty
1	SP1584497	KlickFix Bracket		1

## T-Nuts for Accessories Modulte Seat

157-4067 Rev

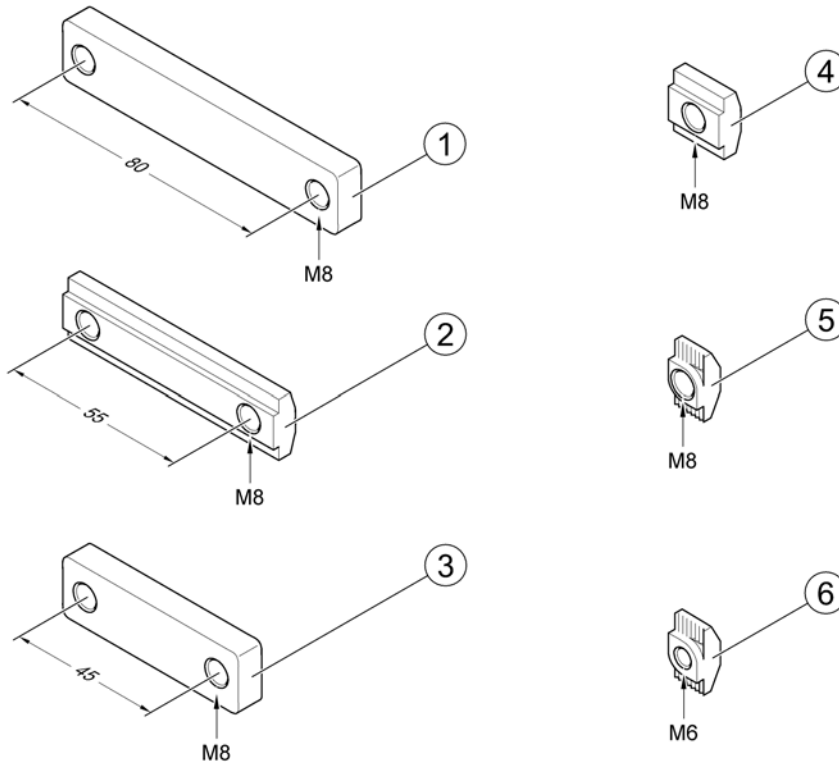

Pos.	Part number	Description	Valid from → until	Qty
1	SP1556401	T-Nut, hole distance 80mm, 2x M8		1
2	SP1572444	T-Nut, hole distance 55mm, 2x M8		1
3	SP1555168	T-Nut, hole distance 45mm, 2x M8		1
4	SP1557341	T-Nut, 1x M8		1
5	SP1573158	T-Nut, 1x M8		1
6	SP1573721	T-Nut, 1x M6		1



Yes, you can.®



---

**Belgium & Luxemburg:**

Invacare nv, Autobaan 22, B-8210 Loppem  
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11  
belgium@invacare.com / www.invacare.be

**Denmark:**

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01  
denmark@invacare.com / www.invacare.dk

**Deutschland:**

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny  
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66  
info@invacare-aquatec.com / www.invacare-aquatec.de

**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica  
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541  
edo@invacare.com / www.invacare.eu.com

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20  
contactsp@invacare.com / www.invacare.es

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com / www.invacare.fr

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland  
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085  
ireland@invacare.com / www.invacare.ie

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34  
italia@invacare.com / www.invacare.it

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede  
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758  
nederland@invacare.com, csede@invacare.com / www.invacare.nl

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo  
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01  
norway@invacare.com, island@invacare.com / www.invacare.no

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee  
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4  
info-austria@invacare.com / www.invacare.at

**Portugal:**

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39  
portugal@invacare.com / www.invacare.pt

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånga  
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08  
sweden@invacare.com, finland@invacare.com / www.invacare.se

**Switzerland:**

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81  
switzerland@invacare.com / www.invacare.ch

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ  
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201  
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220  
uk@invacare.com / www.invacare.co.uk