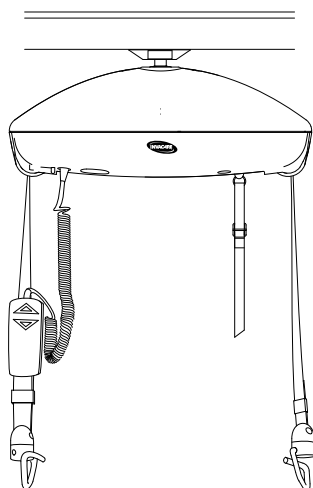
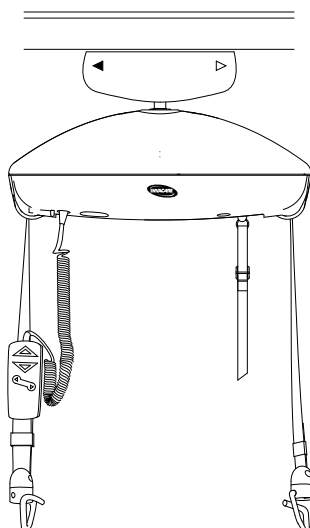
Invacare® Robin®

Robin® , Robin® Mover

Robin®

Robin® Mover

en	Ceiling Hoist User Manual	3
da	Løftemotor Brugsanvisning	25
fr	Lève-personne sur rail Manuel d'utilisation	47
nl	Plafondlift Gebruiksaanwijzing	71
no	Takmontert løfter Bruksanvisning	93
sv	Taklyft Bruksanvisning	115

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

Yes, you can.®

©2019 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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Contents

This manual MUST be given to the user of the product.
BEFORE using this product, this manual MUST be read and saved
for future reference.

1 General	4
1.1 Introduction	4
1.2 Symbols in this document	4
1.3 Service Life	4
1.4 Limitation of Liability	4
1.5 Warranty Information	4
1.6 Compliance	4
1.6.1 Product-specific standards	4
2 Safety	5
2.1 General safety information	5
2.2 Safety information on accessories	6
2.3 Safety information on electromagnetic interference	6
2.4 Labels and symbols on the product	6
2.4.1 Label location	6
2.4.2 Identification label	7
2.4.3 Other labels and symbols	7
3 Product Overview	8
3.1 Intended Use	8
3.2 Main parts of the ceiling hoist	8
3.3 Accessories	8
4 Setup	9
4.1 Included items	9
4.2 Before Use	9
4.3 Installing the ceiling hoist	9
4.4 Inserting hand control plug	9
5 Usage	10
5.1 General safety information	10
5.2 General operating information	10
5.3 Operating the ceiling hoist	10
5.4 Indicator light and audio signal	10
5.5 Charging the Battery	10
5.6 Using the external back-up battery	11
5.7 Emergency stop	11
5.8 Emergency lowering	11
5.9 Resetting the emergency stop or lowering feature	12
6 Patient Transfer	13
6.1 General safety information	13
6.2 Attaching the Sling	13
6.3 Lifting Information	14
6.4 Lifting to and from a seated position	14
6.5 Lifting to and from a lying position	14
7 Maintenance	15
7.1 General maintenance information	15
7.2 Daily inspections	15
7.3 Cleaning and Disinfection	15
7.3.1 General safety information	15
7.3.2 Cleaning Intervals	15
7.3.3 Cleaning instructions	15
7.3.4 Disinfection instructions	15
7.4 Service interval	16
7.4.1 LOLER Statement	16
7.5 Battery Maintenance	16
8 After Use	17
8.1 Transport and Storage	17
8.2 Dismounting the ceiling hoist	17
8.3 Disposal	17
8.4 Reconditioning	17
9 Troubleshooting	18
9.1 Troubleshooting Table	18
10 Technical Data	20
10.1 Dimensions and weight	20
10.2 Electrical System	20
10.3 Environmental conditions	20
11 Electromagnetic compatibility (EMC)	21
11.1 General EMC information	21
11.2 Electromagnetic emission	21
11.3 Electromagnetic Immunity	21
11.4 Test specifications for immunity to RF wireless communications equipment	22

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific sales documents.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare distributor. See addresses at the end of this document.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Symbols in this document

Symbols and signal words are used in this document and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

1.3 Service Life

The expected service life of this product is ten years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.4 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications
- Unauthorised modifications and/or use of unsuitable spare parts

1.5 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.6 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

1.6.1 Product-specific standards

The product has been tested and conforms to ISO 10535 (Hoists for the transfer of disabled persons) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

2 Safety

2.1 General safety information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section.

WARNING!

Risk of injury or damage

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, Invacare provider or qualified technician before attempting to use this product.
- Do not make any unauthorized alterations or modifications to the product.

WARNING!

The maximum safe working load must not be exceeded

- Do not exceed the maximum safe working load of this product or used accessories like slings, spreader bars etc. See documentation or the labeling for the stated maximum safe working load.
- The component with the lowest load limit determines the maximum safe working load of the entire system.

WARNING!

Risk of injury or damage

- Improper use of this product may cause injury or damage.
- Do not attempt any transfer without approval of the patient's healthcare professional.
 - Read the instructions in this user manual and observe trained personnel performing transfer procedures. Then practice transfers under supervision and with a capable person acting as a patient.
 - Special care must be taken with people with disabilities who cannot cooperate while being transferred.
 - The ceiling hoist is advised only to be used to lift or move a patient.
 - The red emergency strap must be adjusted to the caregiver's reach, and must not be removed.
 - If the hoist fails, lower patient onto a suitable surface using the red emergency strap before continuing the transfer by a different method.

WARNING!

Risk of injury or damage

- Inappropriate handling of cables can cause electrical shock and product failure.
- Do not kink, shear or otherwise damage the cables of the product.
 - Make sure that no cables are jammed or damaged, when the product is used.
 - Make sure the wiring is correct and connections are proper.
 - Do not use unauthorized equipment.

WARNING!

Risk of injury or damage

- Excessive moisture will damage the product and may cause electrical shock.
- The patient lift can be used in a bath or shower area but must NOT be used under the shower. The patient must be transferred to a shower chair or use other means for showering.
 - If the patient lift is used in a moisture environment, ensure that the patient lift is wiped clean of any moisture after use.
 - Do not charge the product in a moisture environment or with wet hands.
 - Do not store the product in a damp area or damp condition.
 - See 10.3 *Environmental conditions*, page 20.

WARNING!

Risk of injury or damage

- Ignition sources can cause burns or fire.
- Patient transfer must be performed with a safety space between the hoist and possible ignition sources (heater, stove, fireplace, etc.)
 - The patient and assistants must not smoke during transfer.
 - The sling must not be placed over heat sources (heater, stove, fireplace, etc.)

WARNING!

Risk of injury or damage

- To avoid injury or damage when operating the product:
- Close supervision is necessary when the product is used near children or pets.
 - Do not let children play with the product.

CAUTION!

Risk of injury or damage

- The product can get hot when exposed to sunlight or other heat sources.
- Do not expose the product to direct sunlight for prolonged periods.
 - Keep the product away from heat sources.

IMPORTANT!

- Accumulation of lint, dust and other dirt can impair the product.
- Keep the product clean.

IMPORTANT!

- The electronics may only be serviced by a qualified technician.
- Do not remove the top cover.

2.2 Safety information on accessories

CAUTION!

Risk of injury

Non-original or incorrect accessories may affect the function and safety of this product.

- Due to regional differences, refer to your local Invacare website or catalog for available accessories or contact your Invacare provider.
- See the manual delivered with the accessory for further information and instructions.
- Use only original accessories for the product in use. Under certain circumstances, the use of slings from other manufacturers is possible. See additional information in this section.
- A risk assessment is always to be carried out by a professional prior to issuing lifting equipment. It is important that the Task, Individual, Load, Environment and Equipment are considered in the risk assessment.
- Always choose the sling design and size according to the patient's weight, size and physical ability whilst considering the type of transfer to be carried out.

CAUTION!

Compatibility of slings with attachment system

Invacare uses a common attachment system based on hooks and loops. Loops on the slings are attached to hooks on the lift. Therefore, suitable slings of other manufacturers can be used on this lift as well.

- Use only slings with loop attachments that are suitable for hooks as attachment points.
- Do not use slings designed for "Keyhole or Clip Attachment Systems" or "Tilting Frame Systems".

2.3 Safety information on electromagnetic interference

WARNING!

Risk of malfunction due to electromagnetic interference

Electromagnetic interference between this product and other electrical equipment can occur and disturb the electrical adjustment functions of this product. To prevent, reduce or eliminate such electromagnetic interference:

- Only use original cables, accessories and spare parts, to not increase electromagnetic emission or reduce electromagnetic immunity of this product.
- Do not use portable radio frequency (RF) communications equipment closer than 30 cm to any part of this product (including cables).
- Do not use this product near active high-frequency surgical equipment and the RF shielded room of a system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.
- If disturbances occur, increase the distance between this product and the other equipment or switch it off.
- Refer to the detailed information and follow the guidance in chapter 11 *Electromagnetic compatibility (EMC)*, page 21.

WARNING!

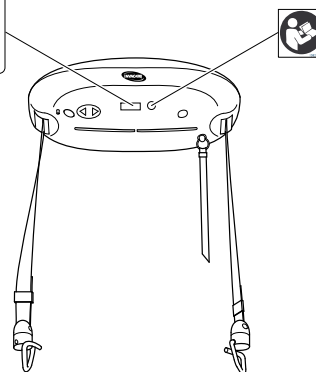
Risk of malfunction

Electromagnetic interference may cause improper operation.

- Do not use this product adjacent to or stacked with other electrical equipment. If such use is necessary this product and the other equipment must be closely observed to verify that they are operating normally.

2.4 Labels and symbols on the product

2.4.1 Label location

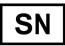
2.4.2 Identification label

The identification label contains the main product information, including technical data.

Symbols

	Serial number
	Reference number
	Manufacturer
	Date of manufacture
	Max. safe working load
	Class II equipment

	Type B applied part
	WEEE conform
	European Conformity
	Medical device

Abbreviations for technical data:

- I_{in} = Incoming Current
- U_{in} = Incoming Voltage
- Int. = Intermittence
- AC = Alternating Current
- Max = maximum
- min = minute

For more information about technical data, see *10 Technical Data, page 20.*

2.4.3 Other labels and symbols

	Refer to User Manual
-----------------------------------------------------------------------------------	----------------------

3 Product Overview

3.1 Intended Use

A ceiling hoist is a battery-powered transfer device, designed to be used indoor in normal temperature and humidity.

If continuously exposed to high humidity, high condensation and corrosives (e.g. chlorine or ammonia gases) the product may be impaired and a reduction in lifetime must be expected.

Common lifting situations are:

- Between the bed and a wheelchair
- To and from the toilet
- Lowering and raising patients to/from the floor

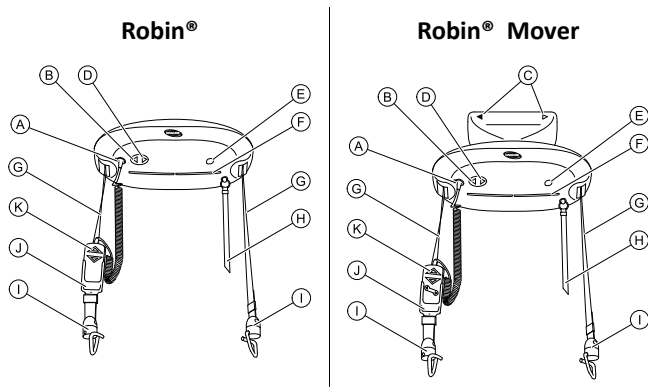
The maximum safe working load is stated in *10 Technical Data, page 20*

The ceiling hoist can be used to transfer and position completely or partially immobile patients in the recommended track system. All position changes are possible without assistance of the patient.

A healthcare professional or private person who has received proper training is the intended operator of this product.

There are no known contraindications for this product.

3.2 Main parts of the ceiling hoist

Ⓐ	Connection for hand control
Ⓑ	Control button UP
Ⓒ	Directional arrows on Robin® Mover
Ⓓ	Control button DOWN
Ⓔ	Connection for external back-up battery
Ⓕ	Indicator light
Ⓖ	Lifting straps
Ⓗ	Red emergency strap
Ⓘ	Strap hooks
Ⓙ	Retainer for hand control (accessory)
Ⓚ	Hand control

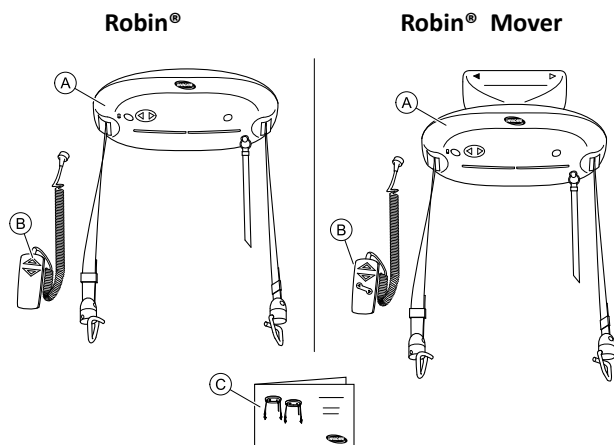
3.3 Accessories

i Due to regional differences, refer to your local Invacare website or catalog for details about available accessories or contact your Invacare provider.

- 2- and 4-point spreader bar including flexiscope version
- Stretcher for horizontal lift
- Scale including spreader bar
- Holder for hand control
- Battery charger
- Charger set (charger and holder for hand control)
- Extension straps and extension cables for hand control
- External back-up battery
- Charger for external back-up battery
- Sling models for “Loop and Coat Hanger Spreader Bar System”:
 - Full body support slings – without head support
 - Full body support slings – with head support
 - Slings for dress/toileting – with or without head support
 - Slings for amputee
 - Slings for walking training

4 Setup

4.1 Included items

- (A) Ceiling hoist including two lifting straps with hooks and an emergency strap. Robin Mover also includes the trolley.
- (B) Hand control
- (C) User manual

4.2 Before Use

CAUTION!

– The ceiling hoist is not intended to be used together with MEDICAL ELECTRICAL EQUIPMENT.

Before first use, all features of the hoist system must be inspected by specialized service technicians.

Perform a visual inspection of the ceiling hoist. If the packaging is damaged upon arrival, the ceiling hoist parts must be thoroughly examined for visible damage or defects. If there is any suspicion of damage, the ceiling hoist must not be used before specialized service technicians have approved it.

The emergency stop is activated during shipment. Reset the emergency stop before first use. See 5.9 *Resetting the emergency stop or lowering feature*, page 12.

It is possible that the hoist's battery is discharged on account of self-discharge. Charge the hoist's battery before first use. See 5.5 *Charging the Battery*, page 10.

4.3 Installing the ceiling hoist

The ceiling hoist has to be mounted to a rail system for use. The rail system has to be installed and approved according to EN 10535 by a specialized dealer or specialized service technician.

Installing Robin®

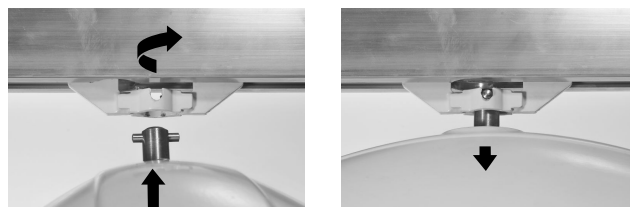
The Robin® requires a trolley, holding the ceiling hoist in the rail system. The trolley has to be pre-installed by a specialized dealer or service technician.

To mount the ceiling hoist to the trolley, do the following:

1. Locate the hoist trolley at the \varnothing 32 mm round recess in the rail.

2. Lift the hoist and push the suspension pin through the trolley opening.
3. Turn the hoist by 90 degrees to fix it in the trolley.


4. Adjust the length of the emergency stop/lowering cord to within the user's reach.

Installing Robin® Mover

Robin® Mover is a ceiling hoist with the trolley already attached. The trolley can therefore not be pre-installed in the rail system.

The ceiling hoist has to be installed by a specialized dealer or service technician.

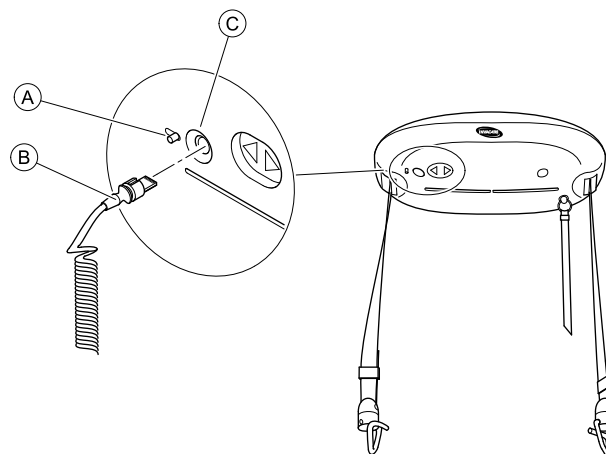4.4 Inserting hand control plug

1. Slide and hold locking lever (A) away from the aperture (C).
A gate in the aperture will open.
2. Make sure the plug (B) is in the correct orientation with the slot upwards (see image). Insert plug into aperture.
3. Release lever, so that the gate closes and hold the plug in the aperture.

Removing the hand control plug

1. Slide and hold locking lever (A) away from the aperture (C).
2. Pull the plug (B) out of the aperture.
3. Release lever.

5 Usage

5.1 General safety information

WARNING!

Risk of injury or damage

Before using the lift with a patient, refer to the following safety information and instructions:

- 2 Safety, page 5
- 6 Patient Transfer, page 13

5.2 General operating information

CAUTION!

If the lifting straps are guided incorrectly in the hoist (e.g. the straps become twisted), the ceiling hoist will automatically turn off.

- Keep the straps straight to ensure they feed into the hoist correctly. The straps are held taught by the weight of the hooks.
- Keep the straps freely suspended when the unloaded hoist is lifting/lowering.

If a user unintentionally traps the hand in the lifting strap during lifting, the hoist will automatically turn off. The hoist stops immediately when a hand gets in contact with the strap inlet.

- Keep your hands away from the strap inlet during lifting.

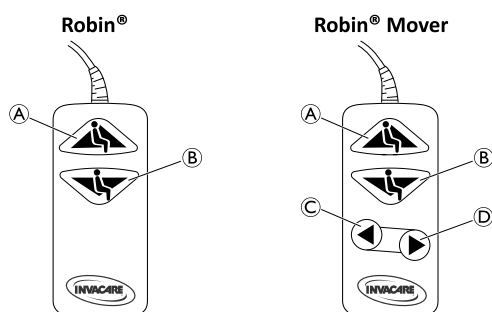
CAUTION!

The hand control is protected against splashed water (IPX4). Extended contact with water will damage the hand control.

- Do not immerse the hand control in liquids, even for a short time.
- Avoid any contact with water or other liquids.

5.3 Operating the ceiling hoist

The ceiling hoist is in operation mode as soon as a button on the hand control is pressed.

Lifting and lowering the straps

When the ceiling hoist is not loaded, it will lift/lower at high speed. When loaded with more than 10 kg, the lifting/lowering speed is automatically reduced to a safe level.

1. To lift the straps, press the UP button (A) on the hand control.
2. To lower the straps, press the DOWN button (B) on the hand control.

The ceiling hoist can also be operated by pressing the UP and DOWN buttons on the underside of the hoist.

Traverse Robin® along the track

1. Use the two lifting straps to pull the ceiling hoist along the track.

DO NOT use the red emergency strap to traverse the ceiling hoist.

Traverse Robin® Mover along the track

1. Perform one of the following:
 - Hoist is loaded: Press button LEFT (C) or RIGHT (D) on the hand control to traverse the ceiling hoist along the track.
 - Hoist is not loaded: Use the two lifting straps to pull the hoist along the track.

DO NOT use the red emergency strap to traverse the ceiling hoist.

It is not possible to activate both traversing and lifting/lowering function simultaneously. When the traversing function is activated, the blue indicator light on the bottom of the hoist is off. See 5.4 Indicator light and audio signal, page 10.

5.4 Indicator light and audio signal

Hoist

Status	Indicator light	Audio signal
Off / Stand by	Off	No
Hoist traversing (Robin® Mover only)		
Hoist in use (Up/Down)	Blue	No
Low battery	Blue Flashing	Yes

The blue indicator light on the underside of the hoist turns off automatically after 30 seconds.

Wall Mounted Charger

Status	Indicator light	Duration
Battery not connected	Yellow	n/a
Initializing and analysis of battery	Yellow	12–14 seconds
Fast charge	Red	5 min — 3 hours
Charged (trickle charge)	Green	n/a
Fault condition (battery defect)	Alternating red and green	n/a

5.5 Charging the Battery

If the charge status becomes low, an audible indicator sounds and the indicator lamp on the bottom of the ceiling

hoist flashes (see also *5.4 Indicator light and audio signal, page 10*). When this happens, there is usually sufficient power to lower the patient.

i It is not possible to use the hoist while the battery is charging.

1. Finish the current lift in progress
If the current lift in process discharges the battery completely, an external backup battery may be used to temporarily power the lift. See *5.6 Using the external back-up battery, page 11*.
2. Move the ceiling hoist to the charging station.
3. Assure that the hand control and charging station are clean and dry before charging
4. Place the hand control into the charging station.
A yellow indicator lamp on the charger indicates the hand control is connected. The indicator lamp illuminates red while charging.
5. When fully charged, the indicator lamp on the charger turns green.
As long as the ceiling hoist is not in use, leave the hand control in the charging station.
6. To use the ceiling hoist, remove the hand control from the charger.

i Keeping the batteries charged guarantees hoist functionality and maintains the battery to ensure a long lifetime.

If the audible indicator beeps and the blue indicator on the hoist flashes after the hoist has been fully charged and only been in use for a short period of time, the battery may need exercise or replacement. See *7.5 Battery Maintenance, page 16*.

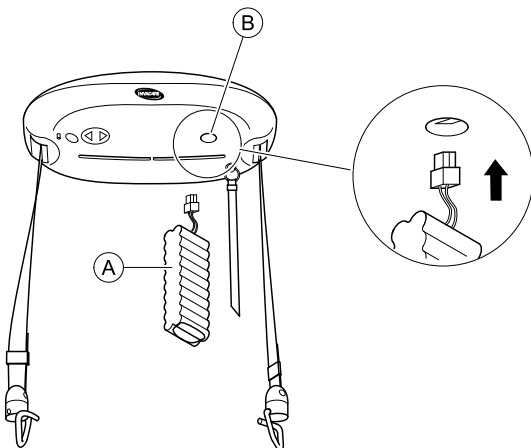
If the ceiling hoist will not be used for more than 4 weeks, Invacare recommends disconnecting the charger and pulling the emergency stop.

The charger is connected to the mains with a mains plug. Make sure that the mains plug is accessible and can be unplugged if required.

5.6 Using the external back-up battery

If the ceiling hoist has to be used immediately with its battery discharged, an external back-up battery can be used.

i Use the external back-up battery only as an exception and when absolutely necessary. The external back-up battery is available as an accessory.


1. Insert the plug of the external back-up battery (A) into aperture (B) underside of the hoist. The external back-up battery should hang freely during patient transfer.
2. Operate the ceiling hoist as described in chapter 5.3 *Operating the ceiling hoist, page 10*.
3. Recharge the external back-up battery in order to have it fully charged for the next usage.

Charging the external battery:

1. Place the plug in the charger socket (a special adapter is necessary).
2. The LED light turns green to indicate the unit is fully charged (approx. charging time 2 hours).

5.7 Emergency stop

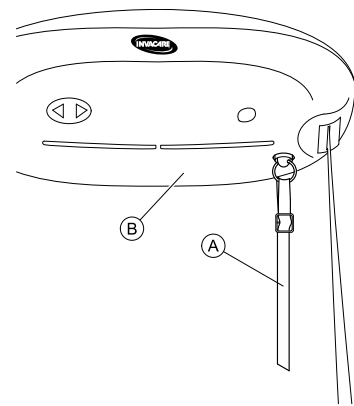The emergency stop function is used to stop the patient from lowering or lifting if the ceiling hoist does not stop or react to the hand control.

1. Pull and release the red emergency strap (A) underside of the hoist (B) to activate the emergency stop. The ceiling hoist stops any lifting or lowering movement immediately.

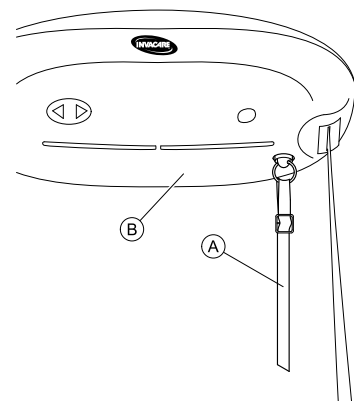
IMPORTANT!


When the red emergency strap has been pulled, the hoist functions, including the end stop, are deactivated. The blue indicator light turns off.
– To activate the hoist functions again, reset the emergency stop. See *5.9 Resetting the emergency stop or lowering feature, page 12*.

5.8 Emergency lowering

The emergency lowering function is used to lower the patient only if the hoist has stopped working.

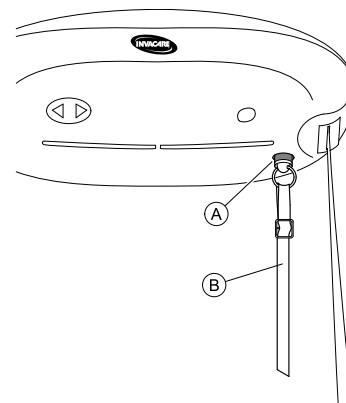1. Pull and hold the red emergency strap **(A)** underside of the hoist **(B)** to lower the patient. Keep on pulling, until the patient is lowered to a safe position.
2. Release the red emergency strap to stop lowering the patient.

! **IMPORTANT!**

- When the red emergency strap has been pulled, the hoist functions are deactivated and the blue indicator light turns off.
 - Pull the red emergency strap only until the patient is lowered to a safe position.
 - To activate the hoist functions again, reset the emergency stop. See 5.9 *Resetting the emergency stop or lowering feature*, page 12.

5.9 Resetting the emergency stop or lowering feature

WARNING!

- If the emergency stop or emergency lowering must be used, there is a problem with the system. Contact your dealer or Invacare representative for service before using the system again.

When the red emergency strap **(B)** has been pulled, the reset button **(A)** pops out so that a red ring is visible. The hoist will not function until the emergency stop or lowering feature is reset.

1. Press the reset button **(A)** to reset the emergency stop or lowering feature.
2. Activate the hand control to resume use of the hoist.

6 Patient Transfer

6.1 General safety information

WARNING!
Risk of Injury or Damage

Improper use of this product may cause injury or damage.

- Before transferring to a stationary object (wheelchair, bed, commode or other surface) check that its weight capacity can withstand the patient's weight.
- If applicable the brakes of the stationary object (wheelchair, bed etc.) must be applied before lowering the patient onto or lifting the patient off.

WARNING!
Risk of Injury or Damage

Damage to parts of the lift caused by impact with the floor, walls or other stationary objects may cause damage to the product and lead to injury.

- Do not allow parts of the lift to impact the floor, walls or other stationary objects.
- Always store the hand control properly when not in use.

WARNING!
Risk of injury

There is a risk of injury to the patient and caregiver caused by dangling hooks or the hand control.

- Always be aware of the position of the hooks and hand control during lifting procedures.

WARNING!
Risk of Entrapment or Strangulation

Items in the patient's surroundings can cause entrapment strangulation during lifting. To avoid entrapment or strangulation:

- Before lifting, check that the patient is completely free of his/her surroundings.
- Do not allow the hand control cord or the lifting straps to become entangled around the patient and caregivers.

WARNING!
Risk of Entrapment

There is a risk of entrapment between the hooks and the sling as well as between the hooks and the hook housing.

- Use caution when lifting.
- Never put hands or fingers on or near the hooks when lifting.
- Ensure the patient's hands and fingers are away from the hooks before lifting.

IMPORTANT!

All transfer procedures described in the following can be performed by one (1) assistant. However, Invacare recommends that the procedures are performed by two (2) assistants whenever possible.

6.2 Attaching the Sling

WARNING!
Risk of Injury

Usage of wrong or damaged slings can cause the patient to fall or cause injury to assistants.

- Use an Invacare approved sling that is recommended by the individual's doctor, nurse or medical assistant for the comfort and safety of the individual being lifted.
- Invacare slings and patient lift accessories are specifically designed to be used in conjunction with Invacare patient lifts.
- After each laundering (in accordance with instructions on the sling), inspect sling(s) for wear, tear, and loose stitching.
- Bleached, torn, cut, frayed, or damaged slings are unsafe and could result in injury. Discard immediately.
- DO NOT alter slings.

WARNING!
Risk of Injury

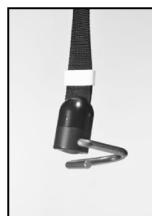
Improperly attached or adjusted slings can cause the patient to fall or cause injury to assistants.

- Be sure to check the sling attachments each time the sling is removed and replaced, to ensure that it is properly attached before the patient is removed from a stationary object (bed, chair or commode).
- DO NOT use any kind of plastic back incontinence pad or seating cushion between patient and sling material that may cause the patient to slide out of the sling during transfer.
- Position the patient in the sling as directed by the instructions provided with the sling.
- Adjustments for patient safety and comfort should be made before moving the patient.

The information in this section is only a general guideline. Refer to the sling user manual for more details about the slings.

A sling with four to six straps designed for mounting on hooks should be used when using the ceiling hoist.

The loops of the sling may be equipped with color coded straps providing different length, to place the patient in various positions. Match the corresponding colors on each side of the sling for an even lift of the patient.

The sling straps are to be hooked in pairs (one from the back and one from the leg) on their own suspension hook.

When attaching the sling to the hooks, and the straps are twisted, the straps will rotate and stretch out, thus ensuring trouble-free operation.

6.3 Lifting Information

WARNING!

Only personnel who have received training regarding the use of lifting equipment and fitting of slings should use the hoist.

- Plan the move. Avoid leaving the client in the sling unattended.
- The ceiling hoist lifts quickly. Before lifting, check that the client is completely free of his/her surroundings.
- The patient's body parts must not be in danger of becoming trapped.
- Check that the red emergency strap, hand control and hand control cable are free of the lifting straps, client and other objects before the ceiling hoist is activated and moved up or down.
- When the ceiling hoist is used correctly, the client should only be lifted to the extent that she/he is clear of the surface and should be moved at this height.

6.4 Lifting to and from a seated position

Follow this procedure when lifting a patient from a seated position (wheelchair, commode, etc.).

1. Traverse the ceiling hoist along the track towards the patient to be lifted.
2. Position the suspension hooks at the same height as the patient's chest and no closer than mid-thigh.
3. Position the suspension hooks so they are parallel to the patient's shoulders.
4. Position the patient in the sling. Refer to the sling user manual.
5. Attach the sling to the suspension hooks. Refer to 6.2 *Attaching the Sling, page 13*.

6.5 Lifting to and from a lying position

Follow this procedure when lifting a patient from a lying position.

1. Traverse the ceiling hoist along the track towards the patient to be lifted.
2. Position the suspension hooks above the patient to be lifted.
3. Align the suspension hooks with the patient's shoulders.
4. Position the patient in the sling. Refer to the sling user manual.
5. Elevate the head of the bed, so that the user is sitting up.
6. Attach the sling to the suspension hooks. Refer to 6.2 *Attaching the Sling, page 13*.

7 Maintenance

7.1 General maintenance information

CAUTION!

Risk of injury or damage

- Do not perform any maintenance or service procedures while the product is in use.

Follow the maintenance procedures described in this manual to keep your product in continuous service.

Further maintenance and inspection procedures, to be performed by a qualified technician, are described in the service manual of this product. Service manuals can be obtained from Invacare.

7.2 Daily inspections

WARNING!

Risk of injury or damage

- Damaged or worn parts can affect the safety of the lift.
- The patient lift should be checked each time it is used.
 - Do not use the lift if damage is found or you question the safety of any part of the lift. Contact your Invacare provider immediately and ensure the lift is not used until repairs are made.

Daily Inspection Checklist

- Visually inspect slings, spreader bar (optional), lifting straps and strap hooks. Check all parts for damage or wear.
- Check the emergency stop and emergency lowering function. Check all parts for external damage or wear.
- Check that all hardware and attachment points for damage or wear. Check all parts for external damage or wear.
- Verify that the hand control is functional.
- Check the battery charger.
- Charge the battery every day the lift is used.

7.3 Cleaning and Disinfection

7.3.1 General safety information

CAUTION!

Risk of contamination

- Take precautions for yourself and use appropriate protective equipment.

CAUTION!

Risk of electric shock and product damage

- Switch off the device and disconnect from mains, if applicable.
- When cleaning electronic components consider their protection class regarding water ingress.
- Make sure that no water splashes to the plug or the wall outlet.
- Do not touch the power socket with wet hands.

IMPORTANT!

- Wrong fluids or methods can harm or damage the product.
- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
 - Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
 - Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
 - Always make sure that the product is completely dried before taking into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

7.3.2 Cleaning Intervals

IMPORTANT!

- Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination. Clean and disinfect the product
- regularly while it is in use,
 - before and after any service procedure,
 - when it has been in contact with any body fluids,
 - before using it for a new user.

7.3.3 Cleaning instructions

IMPORTANT!

- The product does not tolerate cleaning in automatic washing systems, with high-pressure cleaning equipment or steam.

Cleaning the lift

Method: Wipe off with a wet cloth or soft brush.

Max. temperature: 40 °C

Solvent/chemicals: Ordinary household cleaning agent and water.

Drying: Wipe dry with a soft cloth.

Cleaning the sling

Refer to the washing instructions on the sling and to the sling manual for cleaning details.

7.3.4 Disinfection instructions

Information on recommended disinfectants and methods can be found on <https://vah-online.de/en/for-users>.

In domestic care

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

In clinical and long term care

Follow your in-house disinfection procedures and only use the disinfectants and methods specified therein.

7.4 Service interval

WARNING!

Risk of injury or damage

Service must be performed only by a qualified technician.

– Contact your Invacare provider for service.

Service must be performed at least every 12 months unless otherwise stated in local requirements.

The frequency of inspection must be increased, if the product is continuously exposed to high humidity, high condensation and corrosives (e.g. chlorine or ammonia gases), to prevent product impairment.

7.4.1 LOLER Statement

The UK Health and Safety Executive's Lifting Operations and Lifting Equipment Regulations 1998, require any equipment that is used in the workplace to lift a load be subject to safety inspection on a six monthly basis. Please refer to the HSE web site for guidance www.hse.gov.uk.

The person responsible for the equipment must ensure adherence to LOLER regulations.

7.5 Battery Maintenance

If the audible indicator beeps and the blue indicator on the hoist flashes after the hoist has been fully charged and only been in use for a short period of time, the battery may need to be exercised.

To enable the battery to charge fully, the battery should be exercised from time to time.

Following a long period of storage, it could also be necessary to exercise the battery.

It is also suggested that the battery is exercised in situations where the hoist is used infrequently, and/or where the hand control is placed in the charger for long periods of time. In this situation, there will always be capacity enough for a few lifts, but we recommend to exercise the battery as soon as possible afterwards.

Exercise of the battery can be done by charging and discharging the hoist completely several times.

1. Charge the battery until the LED turns green.
2. Remove the hand control from the charger.
3. Raise and lower the straps with a load for one minute.
4. Wait ten minutes.
5. Repeat Steps 3–4 until the battery is completely discharged.

Continue this process after the audible alert sounds and the blue light flashes, indicating the battery is low.

6. Repeat Steps 1–5 two to three times.

8 After Use

8.1 Transport and Storage

Invacare recommends that the ceiling hoist is always transported and stored in the original packaging. Place the ceiling hoist on a soft surface, such as a cloth or foam rubber mat.

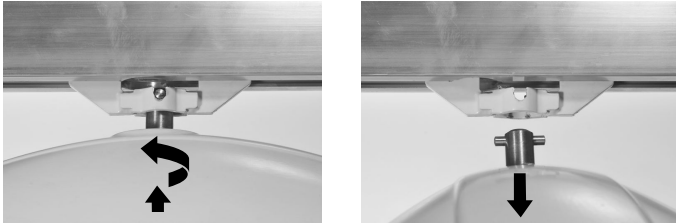
For long term storage, the emergency stop must be activated. This will reduce the discharge of the battery. Refer to 5.7 *Emergency stop, page 11*.

For transport and storage conditions of the ceiling hoist, refer to 10.3 *Environmental conditions, page 20*.

8.2 Dismounting the ceiling hoist

The ceiling hoist can be dismounted for transport or storage.

Robin®

1. Lift the hoist slightly and push the suspension pin out of its holder in the trolley.
2. Turn the hoist by 90 degrees and pull it out of the trolley.

Robin® Mover

Contact your Invacare representative or specialized dealer to have the ceiling hoist dismounted by a specialized service technician.

8.3 Disposal

WARNING!

Environmental Hazard

Device contains batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste.
- Batteries MUST be taken to a proper disposal site. The return is required by law and free of charge.
- Do only dispose discharged batteries.
- Cover terminals of lithium batteries prior to disposal.
- For information on the battery type see battery label or chapter 10 *Technical Data, page 20*.

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

8.4 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see 7 *Maintenance, page 15* and the service manual for this product.

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

9 Troubleshooting


9.1 Troubleshooting Table

SYMPTOMS	PROBLEM	SOLUTION
Ceiling hoist does not respond to the hand control's buttons and up/down buttons on the ceiling hoist are not responding.	Emergency stop activated	Check that the emergency stop is not activated. Refer to <i>5.9 Resetting the emergency stop or lowering feature, page 12.</i>
	System does not have power or battery is discharged	Charge the battery or use emergency battery. Refer to <i>5.5 Charging the Battery, page 10.</i>
	Fuse is blown	The fuse needs to be replaced. Contact an Invacare representative or specialized dealer for service.
Ceiling hoist does not respond to the hand control's buttons, but up/down buttons on the ceiling hoist are responding.	Hand control not connected properly	Check the hand control plug. Refer to <i>4.4 Inserting hand control plug, page 9.</i>
	Hand control defective	Replace the hand control.
Lifting straps will only move down and not up.	Battery nearly discharged	Charge the battery or use emergency battery. Refer to <i>5.5 Charging the Battery, page 10.</i>
	Load is too high	Reduce the load. The hoist is equipped with an over-current safety system.
Lifting straps will only move up and not down.	Hand control defective	Replace the hand control.
	Loose strap function activated	Contact an Invacare representative or specialized dealer for service.
Ceiling hoist speed is the same with and without load.	Non-load quick movement function activated permanently	Contact an Invacare representative or specialized dealer for service.
Lifting straps are twisted and do not straighten out.	Lifting hooks do not turn freely enough	Clean or replace the hooks. Contact an Invacare representative or specialized dealer for service.
	Straps are frayed	Contact an Invacare representative or specialized dealer to replace the straps.
Ceiling hoist does not traverse the rails.	Rails require service or cleaning	Contact an Invacare representative or specialized dealer for service.
	Trolley worn or damaged (Robin® only)	Trolley must be replaced. Contact an Invacare representative or specialized dealer for service.
	Driving wheels worn or damaged (Robin® Mover only)	Driving wheels must be adjusted. Contact an Invacare representative or specialized dealer for service.
	Hand control defective (Robin® Mover only)	Replace the hand control.
Ceiling hoist emits excessive noise when activated.	Bearings, gearwheels or motor defective	The hoist must be replaced. Contact an Invacare representative or specialized dealer for service.
Ceiling hoist will not charge	System has no power	Check that the power supply is switched on and connected to the charging station.
		Contact an Invacare representative or specialized dealer for service.
No indicator light on hoist	System has no power or battery is discharged	Check that the ceiling hoist has power supply and that the battery is recharged.
		Place the hand control in the charging station and recharge the ceiling hoist.
The emergency strap does not turn ceiling hoist off	Ceiling hoist requires service.	Contact an Invacare representative or specialized dealer for service.
The emergency strap does not lower the ceiling hoist.	Ceiling hoist requires service.	Contact an Invacare representative or specialized dealer for service.

 If problems are not remedied by the suggested means, contact an Invacare representative or specialized dealer for service.

10 Technical Data

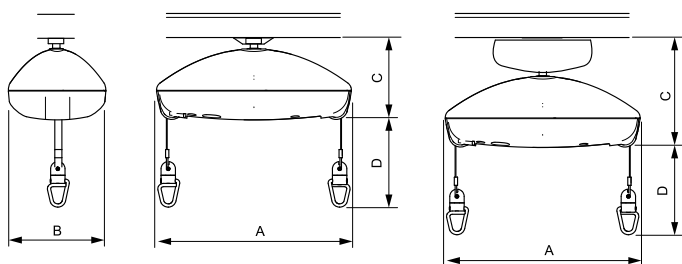10.1 Dimensions and weight

Dimensions

	Robin®	Robin® Mover
Hoist length (A)	470 mm	470 mm
Hoist width (B)	250 mm	250 mm
Hoist height (C)	203 mm	266 mm
Maximum lifting range (D)	2.5 m	2.5 m

Weights

	Robin®	Robin® Mover
Maximum lifting capacity (Safe Working Load)	200 kg	200 kg
Total weight without sling	7.3 kg	10 kg

10.2 Electrical System

	Robin®	Robin® Mover
Maximum current input	0.9 A	
Protection class ¹	Entire device: IPx4 (ceiling hoist: IP24; hand control: IP44)	
Insulation class	Class II equipment	
	Type B applied part	
	Applied part complying with the specified requirements for protection against electrical shock according to IEC60601-1.	
Sound level	50 – 55 dB (A)	
Number of lifts per charge	65 lifts of 0.5 m with 80 kg	

	Robin®	Robin® Mover
Intermittent (periodic motor operation)	10%, max. 2 min. / 18 min.	
Battery	NiMH Battery 24 V / 1.9 Ah	
Charging time	Up to 3 hours at 20°C	
Lifting speed with 200 kg load	2.1 cm/s	2.1 cm/s
Traverse speed in the tracks	n/a	14 cm/s

Charger

Voltage output	During charging: 25 – 36 V DC No-load: 41 V DC
Voltage supply	100 – 240 V AC, 50 – 60 Hz
Protection class ¹	IP41 or IP67 (depending on version)

¹ See product label and label on each electric device for correct protection class. The lowest IP classification decides the overall classification of the device.

- IPx4: Protected against water splashed from any direction.
- IP24: Protected against objects larger than 12.5 mm and protected against water splashed from any direction.
- IP44: Protected against objects larger than 1 mm and protected against water splashed from any direction.
- IP41: Protected against objects larger than 1 mm and protected against vertically dripping water.
- IP67: Dust-tight and protected against temporary immersion into water.

10.3 Environmental conditions

	Storage and transportation	Operation
Temperature	-30 °C to +50 °C	+5 °C to +40 °C
Relative humidity	10 % to 70 %	20 % to 90 % at 30 °C, not condensing
Atmospheric pressure	700 hPa to 1060 hPa	

Allow the product to reach operation temperature before usage:

- Warming up from minimum storage temperature might take up to 24 hours.
- Cooling down from maximum storage temperature might take up to 24 hours.

11 Electromagnetic compatibility (EMC)

11.1 General EMC information

Medical Electrical Equipment needs to be installed and used according to the EMC information in this manual.

This product has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2 for Class B equipment.

Portable and mobile RF communications equipment can affect the operation of this product.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standard. To determine if the emission from this product is causing the interference, run and stop running this product. If the interference with the other device operation stops, then this product is causing the interference. In such rare cases, interference may be reduced or corrected by the following:

- Reposition, relocate, or increase the separation between the devices.

11.2 Electromagnetic emission

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group I	This product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This product is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	Complies	

11.3 Electromagnetic Immunity

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines; 100 kHz repetition frequency ± 1 kV for input / output lines; 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line to line ± 2 kV line to earth	Mains power quality should be that of a typical commercial or hospital environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 0% U_T for 0,5 cycle at 45° steps 0% U_T for 1 cycles 70% U_T for 25 / 30 cycles < 5% U_T for 250 / 300 cycles	Mains power quality should be that of a typical commercial or hospital environment. If the user of this product requires continued operation during power mains interruptions, it is recommended that the product is powered from an un-interruptible power supply or a battery. U_T is the a. c. mains voltage prior to application of the test level.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 V 150 kHz to 80 Mhz 6 V in ISM & amateur radio bands	Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which this product is used exceeds the applicable RF compliance level above, this product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating this product. Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	10 V/m 80 Mhz to 2,7 GHz 385 MHz - 5785 MHz test specifications for immunity to RF wireless communication equipment refer to table 9 of IEC 60601-1-2:2014	Portable and mobile RF communications equipment should be used no closer than 30 cm to any part of this product including cables.

 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

11.4 Test specifications for immunity to RF wireless communications equipment

IEC 60601-1-2:2014 — Table 9

Test Frequency (MHz)	Band ^{a)} (MHz)	Service ^{a)}	Modulation ^{b)}	Maximum power (W)	Distance (m)	Immunity test level (V/m)
385	380 - 390	TETRA 400	Pulse modulation ^{b)} 18 Hz	1.8	0.3	27
450	430 - 470	GMRS 460, FRS 460	FM ^{c)} ± 5 kHz deviation 1 kHz sine	2	0.3	28
710 745 788	704 - 787	LTE Band 13, 17	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9
810 870 930	800 - 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation ^{b)} 18 Hz	2	0.3	28

Test Frequency (MHz)	Band ^{a)} (MHz)	Service ^{a)}	Modulation ^{b)}	Maximum power (W)	Distance (m)	Immunity test level (V/m)
1720 1845 1970	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation ^{b)} 217 Hz	2	0.3	28
2450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^{b)} 217 Hz	2	0.3	28
5240 5500 5785	5100 - 5800	WLAN 802.11 a/n	Pulse modulation ^{b)} 217 Hz	0.2	0.3	9

 If necessary to achieve the immunity test level, the distance between the transmitting antenna and the Medical Electrical Equipment or System may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

^{a)} For some services, only the uplink frequencies are included.

^{b)} The carrier shall be modulated using a 50 % duty cycle square wave signal.

^{c)} As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

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Invacare Ireland Ltd,
Invacare AB