VICAIR[®]

the best in body support

LIBERTY BACK SYSTEM

GB

User manual

NL

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D

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N

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S

Bruksanvisning

DK

Brugermanual

SF

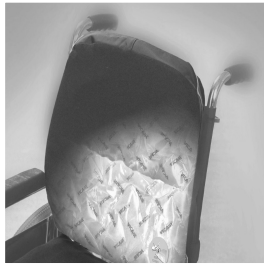
Käyttöohje

F

Mode d'emploi

CE

AVAILABLE STANDARD MODEL

Liberty BC

AVAILABLE STANDARD SIZES

LBS	Minimum and maximum distance between the back tubes:	
	CM	INCH
1	36 - 40	14¼ - 16
2	40 - 44	16 - 17½
3	44 - 48	17½ - 19

fig. A

fig. B

fig. C

fig. D

INTRODUCTION

Your Vicair® Liberty Back System is designed to be used as described in this manual. Please read the whole manual before use.

The system consists of the following parts:

Back strap

The back strap is fitted over the wheelchair side posts and is tensioned with Velcro straps. The back strap replaces the original back support, and allows the cushion to be fitted over it. The back strap is made in different sizes to suit the dimensions of the wheelchair and the cushion.

Inner cushion

The inner cushion is made of a polyvinyl bag, filled with a specific number of triangular, air filled SmartCells™ and a small amount of lubricant. The cushion has two compartments, with a valve fitted in the right rear corner.

Outer cover

The outer cover is made of nylon and Comfair, a breathable fabric which is two-way stretchable. A foam insert is fitted in the back. The large panel on the rear side allows fitting the cushion over the back strap.

ASSEMBLY

Vicair recommends assembly, installation and adjustment of the Liberty Back System under the direct supervision of your therapist or dealer.

Checking the size

Together with your therapist or dealer, make sure the cushion fits the dimensions of your body and of your wheelchair. The cushion should match the width of the wheelchair seat, or be a little smaller.

Fitting the back strap

- Remove the original back support from the wheelchair.
- Open the two small Velcro straps at the bottom corners.
- Slide the side pieces over the backposts of the wheelchair.
- Fasten the two Velcro straps at the bottom corners (fig. A).
- Insert the clamps through the holes in the top corners. Use the small clamps for backposts size 22-25 mm. Use the larger clamps for backposts size 25-28 mm.
- Close the clamps until they fit snugly over the backposts (fig. B).
- Tension the wide Velcro back straps. For added security, the Velcro straps can be threaded through the buckles, so that they are securely locked in position.

Fitting the cushion over the back strap

- Remove the clear plastic wrapper from the cushion.
- Pull the back panel loose from the Velcro fastener at the bottom of the cushion.
- Place the cushion against the back strap and fit the back panel over the back strap (fig. C). Close the Velcro fastener at the bottom of the panel. The finished result should look like fig. D.

INSTALLATION

- Sit in the wheelchair for a few minutes, to allow the back cushion to adapt to your body shape.
- With the aid of your therapist or dealer, adjust the back system to the desired position.
- Unlike the Liberty seating cushions, the valve in the back cushion must NOT be opened to adjust the cushion.

Checking for “bottoming out”

Your back and spine should be supported by the air cells within the cushion; they must not touch the bottom of the cushion. Checking for “bottoming out” should be done during first use and after any adjustments have been made.

- Sit with your back against the cushion for approximately 20 minutes.
- Lean slightly forward in your wheelchair.
- Let your therapist or dealer slide a hand between the cushion and your back.
- Make sure your back is supported by at least one full layer of SmartCells™.
- Do not use the cushion if bottoming out occurs. In most cases, using a different model or size of cushion will solve the problem.

Stability

Using the Liberty Back System may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or dealer.

USE

Checking for redness of the skin

Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, stop using the cushion. Consult your therapist or dealer.

Use at high altitude

Because each SmartCell™ is filled with air, the cushion may behave differently when used at high altitude or in an aeroplane. Vicair recommends not to use the cushion above 2400 m (8000 ft) ASL (above sea level).

Use in motor vehicles

No testing data are currently available about the safe use of Vicair cushions in motor vehicles. Therefore we do not encourage using the cushion in motor vehicles.

Transferring the cushion to another user

Before allowing a second person to use the cushion, clean the inner cushion and wash the outer cover. Disinfect both parts. Make sure the cushion is correctly adjusted to the second person.

Replacing the outer cover

Under normal use, Vicair recommends replacement of the outer cover every year. Replacement covers can be purchased separately.

Flammability

Do not use or leave the cushion close to a naked flame or a high temperature heat source.

MAINTENANCE AND CLEANING

 **Caution:** Do not steam clean or autoclave the cushion. The cushion cannot withstand excessive heat or pressure.

Checking for wear and deflated cells

Every three months, check the cushion for wear and deflated cells. Check all mechanical parts for wear and make sure that all fasteners are secure. In case of serious damage, or a large number of deflated cells, stop using the cushion. Consult your dealer.

Cleaning the inner cushion

- Wipe the inner cushion with a damp cloth using a mild detergent.
- Do not use aggressive or abrasive cleaners, or sharp objects to clean the inner cushion.
- Do not wash the inner cushion in a washing machine.

Cleaning the outer cover

Remove the foam insert. The outer cover can then be machine washed inside-out. See the instructions on the label attached to the inside of the cover.

Disinfecting the cushion

In case of soiling with body fluids, and when transferring the cushion to another user, the cushion must be disinfected to avoid cross contamination.

- Clean the inner cushion and the outer cover as described above. When washing the outer cover, preferably use an anti-bacterial agent.
- Spray disinfectant on the outer cover and the inner cushion.

WARRANTY

Vicair offers a limited warranty against defects in workmanship and materials for a period of two (2) years from the original date of purchase, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning or negligence is excluded from this warranty. Should a defect in materials or workmanship occur within two (2) years from the original date of purchase, Vicair will, at its own choice, repair or replace the product free of charge. The outer cover is excluded from this warranty.

Remedies for breach of express warranties herein are limited to repair or replacement of the product. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

For additional product information, see our website: www.vicair.com.

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