
# Invacare® Verso™ II

Alegio™ NG, SB® 755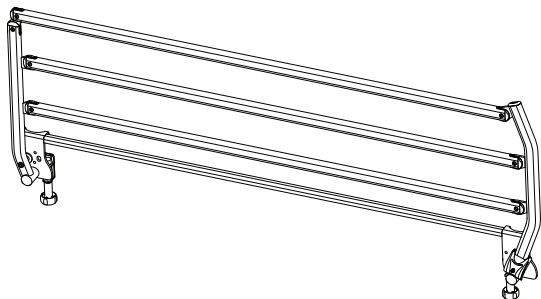

en	Bed side rail User Manual .....	1
es	Barandilla para cama Manual del usuario .....	4
fr	Barrière latérale du lit Manuel d'utilisation.....	7
it	Sponda laterale del letto Manuale d'uso .....	11
nl	Zijhek voor bed Gebruiksaanwijzing .....	14
pt	Guarda lateral da cama Manual de utilização.....	18
sv	Sänggrind Bruksanvisning.....	21

## 1 General

### 1.1 About this manual

This manual belongs to an Invacare accessory and it contains important information about handling and assembly. In order to ensure safety when using the product, read the user manual of the related product carefully and follow the safety instructions.

#### Symbols in this manual

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

#### WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

#### CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

#### IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

#### Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

### 1.2 Intended use

The side rail is intended:

- to prevent the user from falling out of the bed.
- to be operated by a third person and not by the user himself.
- for home care and long term care facilities only.

#### WARNING!

- Any other or incorrect use could lead to hazardous situations.
- *Invacare®* accepts no liability for any use, change or assembly of the product other than as stated in this User manual.

### 1.3 Service Life

The expected service life of this product is ten years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

### 1.4 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

### 1.5 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I. The launch date of this product is stated in the CE declaration of conformity.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

This manual **MUST** be given to the user of the product. **BEFORE** using this product, this manual **MUST** be read and saved for future reference.

**Yes, you can.®**

## Product-specific standards

The product has been tested and conforms to IEC 60601-2-52 (Particular requirements for the basic safety and essential performance of medical beds) and all related standards.

For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

## 2 Safety

### 2.1 General warnings

**WARNING!**

**Risk of entrapment / suffocation**

There's a risk of entrapment / suffocation between mattress support, side rail and bed end or between moving parts and objects placed nearby the bed.

- The product must not be used by persons under 12 years of age, or by persons with a body size equivalent to an average 12 years old or smaller.
- The bed, in combination with side rails must not be used by persons having a physical size less than 146 cm, a weight less than 40 kg or a body mass index (BMI) less than 17.
- Due to mattress compression, an increased risk may occur over time. Periodically monitor gaps between the bed, mattress and/or side rail. Replace mattress if the gaps may lead to entrapment.

**WARNING!**

**Risk of user slipping through the openings**

The bed fulfils all requirements regarding maximum distances. However, it is possible that users with small body dimensions slip through the openings between the side rails or through the opening between the side rail and the mattress support.

- Pay special attention, if the bed is used for the care of users with small body dimensions.

**CAUTION!**

**Risk of injury**

There is a risk of entrapment while operating the side rail.

- Pay attention to your fingers.
- Never force or drop the side rail while handling it.

### 2.2 Mattresses

**WARNING!**

**Safety aspects regarding combination of side rails and mattresses:**

In order to obtain the highest possible safety level, when using side rails on the bed, it is important to respect the minimum and maximum measures for mattresses.

- For correct mattress measures; see Bed User Manual for the bed in use.

**WARNING!**

**Risk of entrapment and/or suffocation**

The user could get trapped and/or suffocate, if the horizontal space, between the mattress and the inside of the side rail, is too big.

- Follow the minimum width (and length) of mattresses in combination with a given side rail, as stated in the mattress table.
- Be aware that using very thick or soft mattresses (low density), or a combination of these, increases the risk.


**WARNING!**

**Risk of falling**

The user can fall over the edge and get seriously injured, if the vertical distance, between the top of the mattress and the edge of the side rail/bed end, is too short.

- Always keep a minimum distance of 22 cm. The maximum mattress height in combination with a given side rail is stated in the mattress table.

## 3 Components

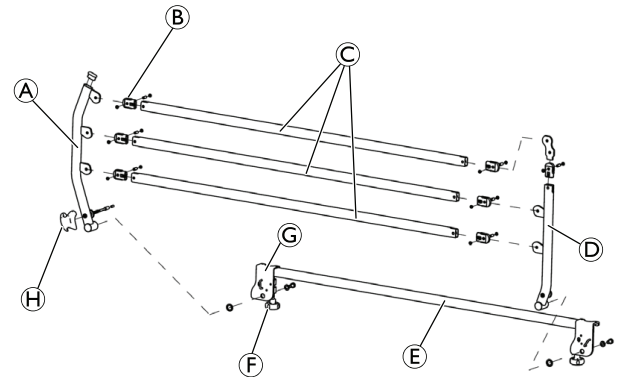- (A) Foot tube left
- (B) Pivot part
- (C) Bar
- (D) Head tube left
- (E) Left base fixation
- (F) Knob
- (G) Jaw
- (H) Locking button

## 4 Accessories

**Important**

- Use only original accessories and spare parts. Spare parts lists and extra user manuals can be ordered from *Invacare*.


For repair, please contact your local Invacare® dealer. Service and repair must be carried out by a properly trained technician.

## 5 Setup

### 5.1 Receiving the product

When you receive the product, check the packaging. If the product shows any signs of damage upon delivery, do not use the product.

#### Scope of delivery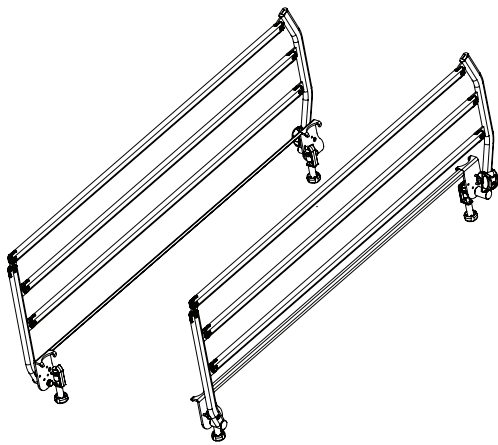


- 1 pair of side rails
- User manual

## 5.2 Installing the side rail


The side rails are provided completely ready to be used and installed on the bed. No tool is necessary to its installation.

1. The installation on the bed must be done with the side rail in locked position.
2. Place both jaws of the side rail on the frame of the bed, by positioning the locking button in footboard. The jaw opposite straddling the lifting pole support to assure a lower space of 60 mm (< 60 mm) between the side rail and the bed end head.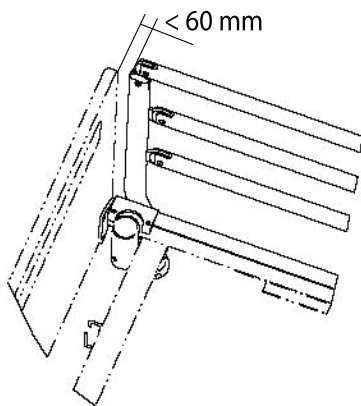


**!** **IMPORTANT!**  
Risk of squeezing or entrapment  
– The distance between the side rail and the head bed end must be smaller than 6 cm.

3. For beds equipped with bed ends with fast locking, it is beforehand necessary to pull and turn  $\frac{1}{4}$  of tour the trigger, previously positioned the jaw at the head of bed under the trigger and relocks it after.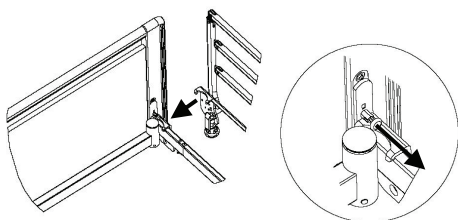

4. Once jaws positioned well on the frame of bed, fix the side rail by using both knobs and tighten them firmly.

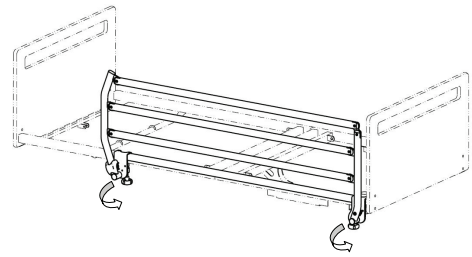### **WARNING!**

#### **Risk of injury or damage**

Not tighten and/or an insufficient tightening of knobs can entail a risk of fall of the side rail and/or of the patient.

- Fix the side rail by using both knobs and tighten them firmly.

5. Make some manipulations to verify the smooth running (folding / unfolding, locking / unbolting, tightening).

## 6 Usage

### 6.1 Operating the side rail

#### To open (fold the side rail):

1. Pull the locking button and start the folding movement with one hand on the upper horizontal bar.
2. Release the locking button during the folding movement and handling the side rail until its lower position (under the mattress).
3. Try moving the side rail up-/downwards to make sure it is properly locked.


### **CAUTION!**

#### **Risk of pinching**

– To avoid all finger pinch out during the manipulation, fingers must be positioned in the two lateral dug areas designed for that purpose, in locking button (grey zone).

#### To close (unfold the side rail):

1. Pull the side rail by the upper bar and rising until it engage in the locking system.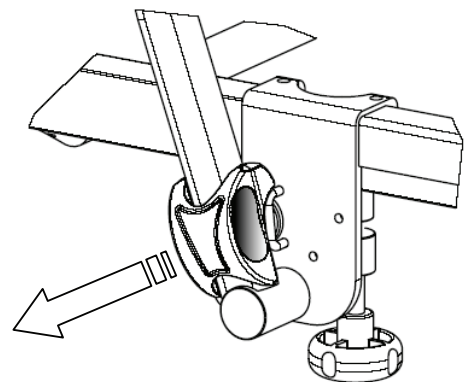

### **WARNING!**

#### **Risk of injury or damage**

Insufficient engaging in the locking system can entail a risk of fall of the side rail and/or of the patient.

- Always verify the good locking of the side rail before leaving unsupervised the unattended patient.

**WARNING!****Risk of squeezing**

There is a risk of getting trapped between the extended tubes on the side rail and the floor or getting squeezed while moving the side rail up and/or down.

- Be careful while lowering the side rail.

## 7 Maintenance

### 7.1 General maintenance information

After 3 months of use, control their fixing, the system of locking and its mechanism. In the event of normal use, the first control of maintenance must be after 2 years then each year.

### 7.2 Cleaning

- Side rails and bed ends should be washed down using a wet sponge, cloth or brush.
- Use ordinary household cleaning agents. Never use acids, alkalines or solvents such as acetone or cellulose thinner.

### 7.3 Waste disposal


Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

Waste disposal must comply with the laws and regulations pertaining waste handling in each country.

## 8 Technical Data

### 8.1 Dimensions and weights

All measurements and angles are stated without tolerances, on a non-extended bed.

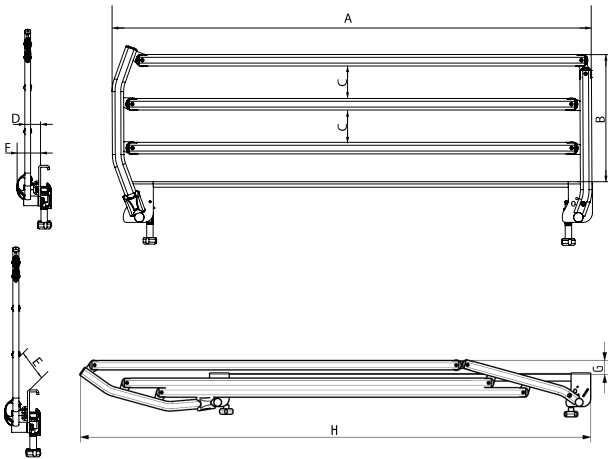Dimensions [mm]		
A	Length	1500
B	Height	400
C	Space between bars	100
D	External dimension out bed	50
E	Space between bed and bar	100
F	External overall dimension out bed	70
G	Height in folding position	55
H	Length in folding position	1940

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Invacare AB
Invacare Limited