Universal High

en	Sling	User Manual	2
da	Sling	Brugsanvisning	3
de	Patientengurte	Gebrauchsanweisung	5
es	Eslinga	Manual del usuario	7
fi	Nostoliina	Käyttöohje	8
fr	Sangles	Manuel d'utilisation	10
it	Imbracatura	Manuale d'uso	12
nl	Liftmat	Gebruiksaanwijzing	13
no	Seil	Bruksanvisning	15
pt	Cesta de transferência	Manual de utilização	16
sv	Lyftselar	Bruksanvisning	18

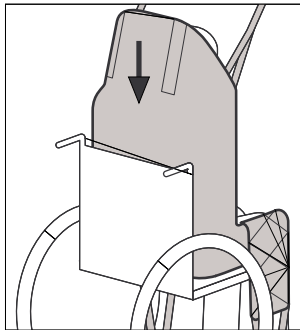
Figure 1

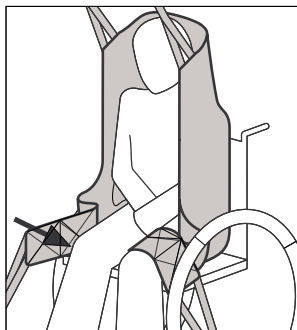
Figure 2

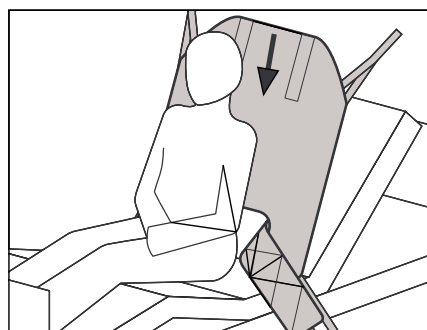
Figure 3

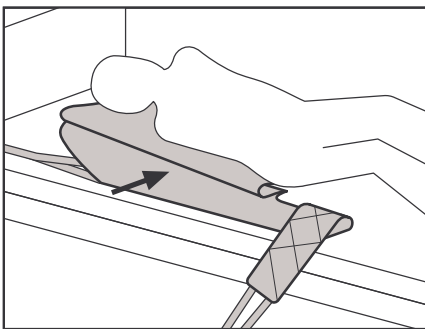
Figure 4

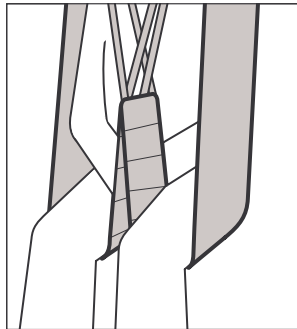
Figure 5

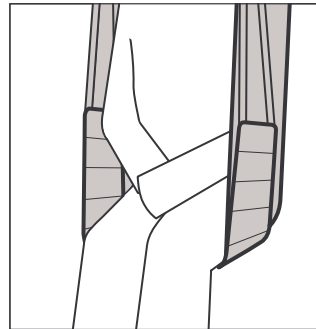
Figure 6

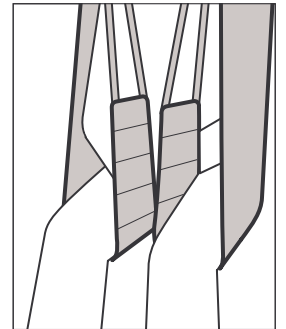
Figure 7

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

General

Symbols

	Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words. WARNING! Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. CAUTION! Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.
	Gives useful tips, recommendations and information for efficient, trouble-free use.
	IMPORTANT indicates a hazardous situation that could result in damage to property if it is not avoided.
	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the EC declaration of conformity.
	Safe Working Load
	Consult the user manual.
	DO NOT exceed 95°C washing temperature.  Remove leg support foam from pockets before washing.
	DO NOT bleach.
	Tumble Dry, Normal, Low Heat
	DO NOT iron.

Intended Use

The Invacare® Universal Sling is available with or without head support, the head support being created by removable plastic spines within the fabric. The sling can be used for highly physically dependent clients and allows for easy access for personal hygiene. It is easily applied in both sitting and lying position.

Service Life

All Invacare slings are guaranteed for one year. Expected service life of the product is 1-5 years. Service life time varies depending on the fabric, frequency of use, laundering practices, and weight carried.

Safety

General Guidelines

WARNING!

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Please check your sling on delivery. If the sling is not in perfect condition, please contact your supplier.

Safe Working Load

	Solid	Net	Spacer	
XS	1483776	1485950	1485955	200 kg
S	1483777	1485951	1485956	200 kg
M	1483778	1485952	1485957	200 kg
L	1483779	1485953	1485958	200 kg
XL	1485949	1485954	1485959	250 kg

Usage

Sling application


WARNING!

- Slings should be checked regularly for wear and damage (in accordance with each country's regulations). NEVER use a damaged sling.
- If you are in any doubt, please do not attempt to use the sling and seek the advice of a professional or your supplier.
- Invacare recommends that a risk assessment is always carried out by a professional prior to issuing lifting equipment. Always choose the sling design and size according to the client's weight, size and physical ability.
- If you are in doubt about the use of Invacare's slings with other manufacturers' equipment, contact Invacare or your supplier for advice.

Sling application — Sitting in chair or wheelchair


1. Lean the client forward ensuring that he is well supported to reduce risk of falling and to promote feelings of security.
2. Slide the sling down behind the back of the client in a central position with the aperture apex aligned with the base of the spine.
3. Sit the client back against the sling. **Figure 1.**
4. Pass the leg support towards the front of the chair to each side of the thigh and then pass each leg support under each thigh. **Figure 2.**

Sling application — Lying in bed or floor

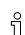 If the client is able to sit up from the bed or floor, this will ease the application.

1. Lean the client forward ensuring that they are well supported to reduce risk of falling and to promote feelings of security.
2. Slide sling down behind the back of the client in a central position with the aperture apex aligned with the base of the spine.

3. Sit the client back against the sling. **Figure 3.**

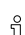 Alternatively, if the client does not have sufficient trunk control, roll the sling to mid-way point and roll the client to one side to position the rolled up sling under the lifted side. Ensure the center of the sling (green mark) is placed centrally along the spine. Roll the client to the other side, unroll the sling and roll the client back into position on the sling. **Figure 4.**

4. One at a time, bend the knees, lift the legs and apply the sling under the legs.

Application of the legs straps

Crossed legs straps

For maximum security, cross the legs straps or use the modesty strap. This will reduce the risk of forward pitching and also maintain a better alignment of the hips and lower limbs. **Figure 5.**

Looping the leg sections together

If the client is an amputee or feels this is more comfortable, each individual leg strap can be placed under both legs. Ensure that the client is not prone to spasm and there has been a risk assessment completed to address the possibility of forward pitching. **Figure 6.**

Uncrossed leg straps

Apply each leg support under each leg, but do not cross the straps. This can be recommended when the client has abdominal sensitivity or requires intimate hygiene. **Figure 7.**

Attach loops to the spreader bar or lifting strap

WARNING!

- Use only CE marked spreader bars and hoists, preferably with a rated Safe Working Load equal to or higher than that of the sling. If the SWL is lower than that of the sling, the SWL of the spreader bar or hoist is applicable.
- Always check that all the straps are properly attached to the spreader bar or lifting strap before attempting to lift the client.

1. Attach the loops to the spreader bar or lifting straps of the lifting equipment.

Attach the shoulder- and head loops before the leg loops.

The sling can be used with spreader bars with 2 attachment points.

For security and comfort reasons, the size (width) of the spreader bar must correspond with the size of the sling.

The position of the client in the sling can be adjusted by using the different adjustment straps. Shorter loops at the shoulders will produce a more vertical lift, which will assist with positioning into a chair or wheelchair. By lowering the loops at the shoulders, a more reclined position can be achieved which is more suitable for a chair to bed transfer.

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©2017 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB