
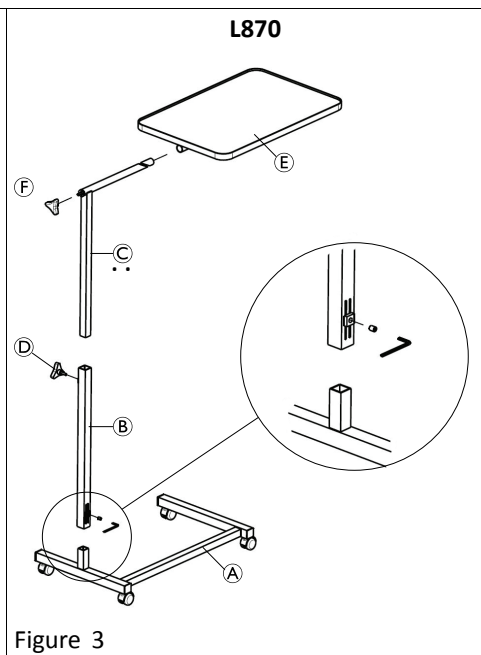
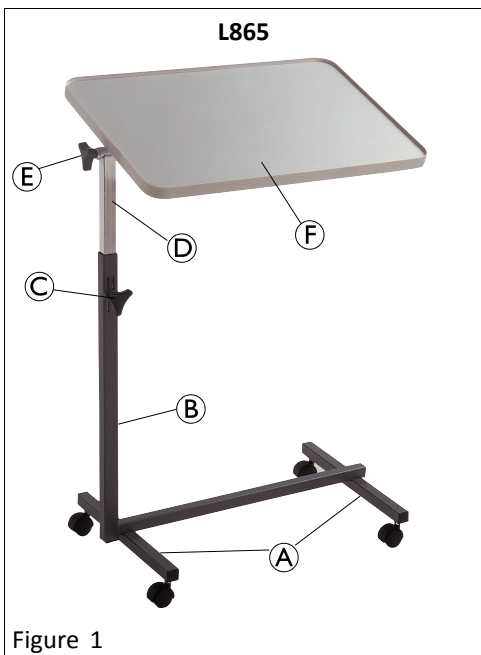
Invacare® Accessories

Invacare® Pausa (L865), Invacare® Kauma (L872), Invacare® Baya (L870)

en	Bed side tables User Manual	2	it	Vassoio letto a carrello Manuale d'uso	6
da	Sengeborde Brugsanvisning	2	lt	Prie lovos statomi staliukai Naudojimosi instrukcija	7
de	Beistelltische Gebrauchsanweisung	3	lv	Galdiņi pie gultas Instrukciju rokasgrāmata	8
es	Mesitas para cama Manual del usuario	4	nl	Bedtafels Gebruiksaanwijzing	8
fi	Vuoteen sivupöydät Käyttöohje	4	no	Sidebord for senger Bruksanvisning	9
fr	Tables de chevet Manuel d'utilisation	5	pt	Mesas de cabeceira Manual de utilização	10
hr	Stolići uz krevet Upute za uporabu	6	sv	Sängbord Bruksanvisning	11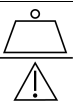

	L865	L872	L870
	10 kg	10 kg	10 kg

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

Safety

CAUTION!

- Check that the product has been delivered in perfect condition before you use it.
- If you are unsure about usage, adjustment or maintenance of the product, contact your Invacare distributor.
- Do not exceed the maximum load, shown on the product label.
- Do not use or store the product outdoors.
- Periodically check stability and rigidity of the product and, where applicable, that the screws are tightened.

Cleaning and disinfection

- Remove any dirt or dust using a soft, damp cloth.
- Use soap and water to remove stubborn stains.
- Dry with a soft cloth.
- For disinfection in hospital environments, follow the hygiene specialist's recommendations.

Warranty information

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located at the end of this manual.

Assembly

Invacare®Pausa (L865)

See figure 1

1. Attach the two feet on castors **A** under the frame **B** using 2 screws.
2. Insert hand screw **C** into the upper section of the frame **B**.
3. Slide the square tube of the tray support **D** into the vertical section of the frame **B**, so that the tray support is located above the frame base.
4. Loosen hand screw **E** and slide the tray **F** on the tray support.
5. Adjust the tray to the desired inclination and tighten hand screw **E**.
6. Adjust the height of the tray and tighten hand screw **C**.

Invacare®Kauma (L872)

See figure 2

CAUTION!

- The height adjustment system is controlled by a spring.
- Follow the instructions in this paragraph carefully and in the indicated order.

1. Install the rack **B** on the base **A**:
 - a. Insert the two large screws in the two-hole plate, and insert the two small cylinders.
 - b. Place this assembly below the castor base and tighten the screws through the base into the rack.
Make sure, the tray is located above the base.
2. Install the tray **C** by sliding it onto its support and fix it using two 8x15 screws.

3. Install the adjustment system:
 - a. Insert the black lever, with the rounded section facing upwards, into the horizontal section and lock it in place (using a 5 x 50 mm bolt and nut).
 - b. Carefully unscrew the small black screw above the red label and replace it with the small grey screw supplied with the table.
DO NOT remove the screw covered by the red label.
4. Remove the protective film from the tray.

Adjusting the height

- To raise the tray, pull it upwards.
- To lower the tray, press the black lever.

Invacare®Baya (L870)

See figure 3

1. Install the square tube **B** on the base **A** using an Allen key.
2. Install the tray **E** on the tray holder tube and lock it with hand screw **F**.
3. Slide the tray holder **C** into the square tube **B**.
Make sure, the tray is located above the base.
4. Adjust the height of the tray and tighten hand screw **D**.

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Invacare AB
Invacare Rea AB