KÜSCHALL ADVANCE

en	Active wheelchair User Manual	4
es	Silla de ruedas activa Manual del usuario	46
pt	Cadeira de rodas activa Manual de utilização	90

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

©2015 Kuschall AG

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Kuschall AG. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Kuschall AG or its subsidiaries unless otherwise noted.

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CONTENTS

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for
future reference.

1	General	5
1.1	Information about the User Manual	5
1.2	Symbols in this manual	5
1.3	Warranty	5
1.4	Standards and Regulations	6
1.5	Intended use	6
1.6	Service life	6
1.7	Copyright protection	6
2	Safety	7
2.1	Safety information	7
2.2	Safety devices	8
2.3	Labels and symbols on the product	8
3	Components and their functions	10
3.1	Overview	10
3.2	Parking brakes	11
3.3	Quick release frame (optional)	12
3.4	Push handles	13
3.5	Side rest, stepless height adjustable, non-locking	14
3.6	Simple armrest, height-adjustable, swivelling, non-locking	15
3.7	Mudguard	16
3.8	Clothes-guard	17
3.9	Footrest	17
3.10	Backrest	17
3.11	Antitipper	18
3.12	Vario-Ax	19
3.13	Tires	19
3.14	Seat cushion	20
3.15	Seat position	20
4	Accessories	21
4.1	Pelvic belt	21
4.2	Transit wheels	22
4.3	Luggage carrier	23
4.4	Air pump	23
4.5	Passive illumination	23
5	Setup	24
5.1	Safety information	24
6	Using the wheelchair	25
6.1	Safety information	25
6.2	Braking during use	26
6.3	Getting in and out of the wheelchair	26
6.4	Driving and steering the wheelchair	27
6.5	Negotiating steps and kerbs	28
6.6	Going up and down stairs	30
6.7	Negotiating ramps and slopes	30
6.8	Stability and balance when seated	32
7	Transport	33
7.1	Safety information	33
7.2	Folding and unfolding the wheelchair	33
7.3	Removing and fitting the rear wheels	34
8	Maintenance	36
8.1	Safety information	36
8.2	Maintenance Schedule	36
8.3	Cleaning	38
8.4	Disinfection	38
9	Troubleshooting	39
9.1	Safety information	39
9.2	Identifying and repairing faults	40

10	After Use	41
10.1	Storage	41
10.2	Re-use	41
10.3	Disposal	41
11	Technical data	42
11.1	Dimensions and weight	42
11.2	Environmental conditions	43
11.3	Materials	43

1 GENERAL

1.1 Information about the User Manual

Thank you for choosing a küschall® wheelchair.

This User Manual contains important information about using the wheelchair. In order to ensure safety when using your wheelchair, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet at www.kuschall.com and enlarge it on-screen as required. If you cannot enlarge the texts and graphics sufficiently, please contact the küschall® distributor for your country;

® addresses on the reverse of this document. If necessary, we will provide you with a high resolution PDF file of the user manual. Moreover, you can have the PDF file read out to you with the aid of suitable programs using special language functions on your computer (e.g. in Adobe®Reader®X: Shift+Ctrl+Y).

The equipment of your wheelchair may differ from the descriptions and diagrams given here due to the wide range of models available.

For latest important product information please refer to our website at www.kuschall.com, your local specialized dealer or contact Invacare® in your country (for addresses, see back page of this manual).

1.2 Symbols in this manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING!

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION!

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT!

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Manufacturer

1.3 Warranty

Küschall AG guarantees that their wheelchairs are free from defects and are fully functional. The warranty covers all faults and defects which are verifiably attributable to faulty construction, substandard materials or poor workmanship. Warranty claims against the manufacturer may only be made by the distributor and not by the user of the product.

The warranty does not cover normal wear and tear, the consequences of improper handling or damage, poor maintenance and incorrect assembly or commissioning by the purchaser or a third person or faults which are attributable

to circumstances beyond our control. Wearing parts are not covered by the guarantee. The warranty is voided if modifications are made to the product which were not authorised and not performed by the specialist dealer or if inappropriate accessories or spare parts are used. The warranty does not cover consequential costs arising from the rectification of defects such as freight and travel expenses, labour costs, fees etc.

The term of the warranty is 24 months. The frame is guaranteed for 5 years.

Furthermore, the terms and conditions are part of the general terms and conditions particular to the individual countries in which the product is sold.

1.4 Standards and Regulations

Quality is fundamental to the company's operation, working within the disciplines of ISO 9001 and ISO 13485.

The wheelchair has been tested in accordance with EN 12183. It includes a test for flammability.

Küschall AG is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We use only REACH and RoHS compliant materials and components.

1.5 Intended use

The active wheelchair is propelled manually and should only be used for independent or assisted transport of a disabled person with mobility difficulties. When used without an assistant, it should only be used by persons who are physically and mentally able to use the wheelchair safely (e.g. to propel themselves, steer, brake). The wheelchair should only be used on level ground and accessible terrain, as well as indoors. This active wheelchair needs to be subscribed and fit to your specific health condition.

WARNING!

Any other or incorrect use could lead to hazardous situations.

1.6 Service life

The expected service life is five years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this manual.

1.7 Copyright protection

This User Manual is protected by copyright. It may not be reprinted or copied or transferred to third parties in part or in full without the prior written consent of the manufacturer.

2 SAFETY

2.1 Safety information

This section contains important safety information for the protection of the wheelchair user and assistant and for safe, trouble-free use of the wheelchair.

WARNING!

Risk of accidents and serious injury

Accidents with resulting serious injury can occur if the wheelchair is wrongly adjusted.

- Adjustments at the wheelchair always must be carried out by a specialist dealer.

WARNING!

Risk due to driving style being unsuitable for the conditions

There is a risk of skidding on wet ground, gravel or uneven terrain.

- Always adjust your speed and driving style to the conditions (weather, surface, individual ability, etc.).

WARNING!

Risk of injury

In a collision you could sustain injury to parts of your body that extend beyond the wheelchair (e.g. feet or hands).

- Avoid an unbraked collision.
- Never drive into an object head-on.
- Drive carefully through narrow passages.

WARNING!

Risk due to wheelchair being out of control

At high speed you could lose control of your wheelchair and overturn.

- Never exceed a speed of 7 km/h.
- Avoid collisions in general.

WARNING!

Risk of developing pressure ulcers

The front edge of the carbon seat plate can cause pressure ulcers at the lower surface of the thigh.

- Always use a seat cushion when sitting in the wheelchair.

WARNING!

Risk of developing pressure ulcers

Due to a wrongly set elastic band area at the rear of the seat plate, an ergonomic seating could be prevented resulting in pressure ulcers.

- Make sure the elastic band area at the seat plate is set correctly, refer to a specialist dealer if necessary.

CAUTION!

Risk of burning

The wheelchair components can heat up when exposed to external sources of heat.

- Do not expose the wheelchair to strong sunlight before use.
- Before usage, check all components that come into contact with your skin for their temperature.

CAUTION!

Risk of getting fingers caught

There is always a risk of getting, e.g. fingers or arms, caught in the moving parts of the wheelchair.

- Make sure when activating the folding or insertion mechanisms of moving parts, such as the removable axle of the rear wheel, folding backrest or antitipper, that nothing becomes caught.

CAUTION!

Risk of cutting your skin

In case of damages the carbon seat fibres can stand out and could injure your skin.

- Check carbon seat plate for damage or outstanding carbon fibres, 9.2 Identifying and repairing faults, page 40 or contact your specialist dealer.

IMPORTANT!

Damage of the seat plate due to missing or insufficiently tensioned elastic band at the rear seat plate.

- Make sure the elastic band is set correctly, refer to a specialist dealer if necessary.

2.2 Safety devices

WARNING!

Risk of accidents

Safety devices that are incorrectly set or no longer working (brakes, antitipper) can cause accidents.

- Always check that the safety devices are working before you use the wheelchair and have them checked regularly by the specialist dealer.

The functions of the safety devices are described in chapter 3 Components and their functions, page 10.

2.3 Labels and symbols on the product

Identification label

The identification label is attached to the frame of your wheelchair and gives the following information:

A	Manufacturer's address
B	Product description

Ⓒ	Date of manufacture
Ⓓ	Instruction to read the User Manual
Ⓔ	Maximum user weight
Ⓕ	Instruction that this wheelchair is not designed for passenger transport in vehicles
Ⓖ	Serial number
Ⓗ	Warning that this is an active wheelchair and can tip


	<p>WARNING Symbol</p> <p>This wheelchair is not intended to be used as a seat in a motor vehicle!</p>
---	--

Antitipper warning label

	<p>Follow user manual instructions prior to the usage of an antitipper device.</p>
--	--

3 COMPONENTS AND THEIR FUNCTIONS

3.1 Overview

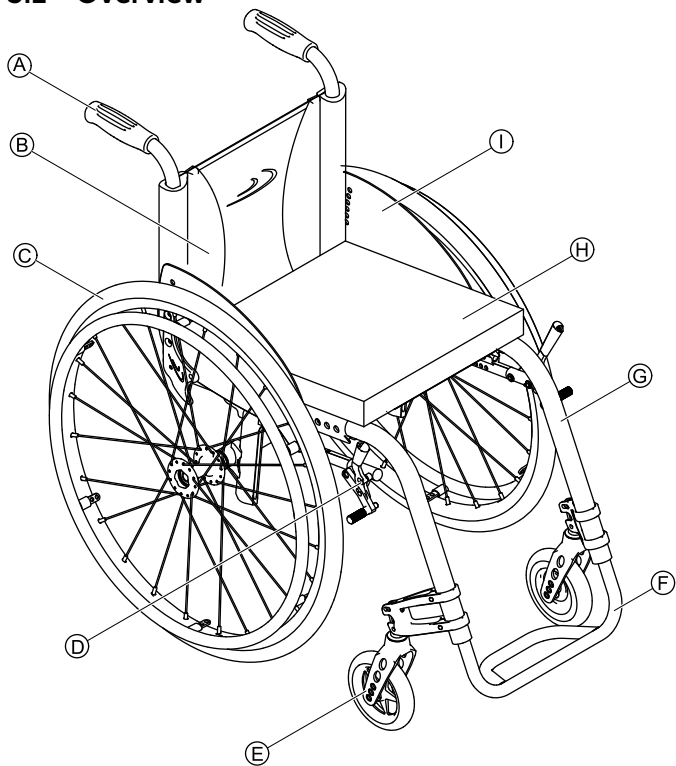Ⓐ	Push handle
Ⓑ	Back

Ⓒ	Rear wheel with handrim
Ⓓ	Parking brake
Ⓔ	Front wheel fork with front wheel
Ⓕ	Footrest
Ⓖ	Frame
Ⓗ	Seat / Cushion
Ⓘ	Clothes-guard

Short description

The küschall Advance® is a rigid frame wheelchair with folding backrest.

The equipment of your wheelchair may differ from the diagram as each wheelchair is manufactured individually to the specifications in the order.

3.2 Parking brakes

The parking brakes are used to immobilise the wheelchair when it is stationary to prevent it from rolling away.

WARNING!

Risk of overturning if you brake sharply

If you apply the parking brakes while you are moving, the direction of movement can become uncontrollable and the wheelchair may stop suddenly, which can lead to a collision or to you falling out.

- Never apply the parking brakes while you are moving.


WARNING!

Risk of overturning

The parking brakes will not operate correctly unless there is sufficient air in the tyres.


- Ensure the correct tyre pressure, 3.13 Tyres, page 19.

Standard brake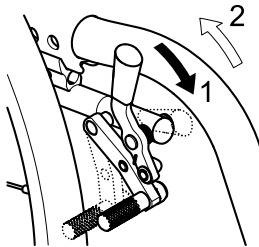


1. To apply the brake, push the brake lever forwards as far as possible.
2. To release the brake, pull the brake lever backwards.

Performance brake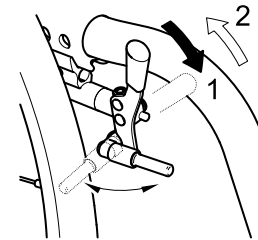


1. To apply the brake, push the brake lever forwards as far as possible.
2. To release the brake, pull the brake lever backwards.

Active brake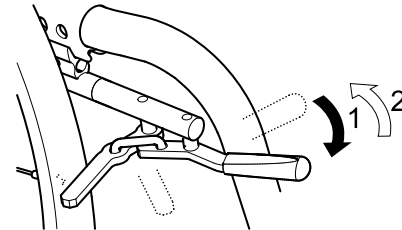

1. To apply the brake, pull the brake lever forwards beside or between your knees as far as possible.
2. To release the brake, push the brake lever backwards beside or between your knees.


Pull brake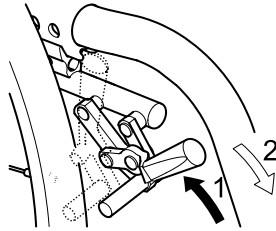

1. To apply the brake, pull the brake lever backwards as far as possible.
2. To release the brake, push the brake lever forwards.

Hub brake (optional)

The hub brake is operated via the brake ring located on the outside of each wheel's hub.

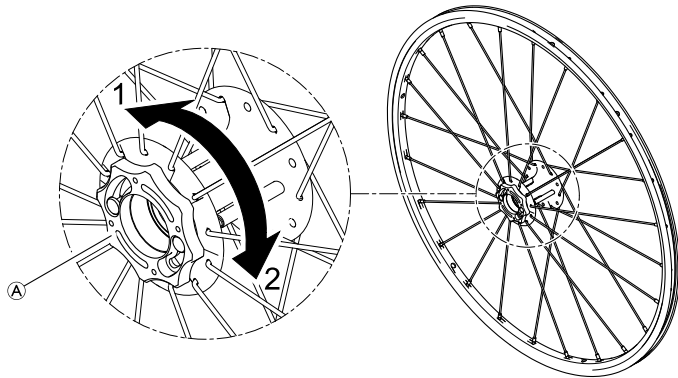1. To activate the hub brake, turn the brake ring **A** in direction 1 until it stops.
2. To release the hub brake, turn the brake ring in direction 2 until it stops.

WARNING!

Risk of overturning if you brake sharply

If you apply the hub brakes while you are moving, the direction of movement can become uncontrollable and the wheelchair may stop suddenly, which can lead to a collision or to you falling out.

- Never apply the hub brake while you are moving.


WARNING!

Risk of injury due to overturning wheelchair

When driving backwards, the wheelchair could overturn due to a snapping in hub brake pin that locks the wheel.

- Contact your specialized dealer immediately, as soon as you hear a rattling noise caused by the hub brake when driving in your wheelchair.
- Avoid driving backwards when you hear such noise until the hub brake is repaired or replaced.

3.3 Quick release frame (optional)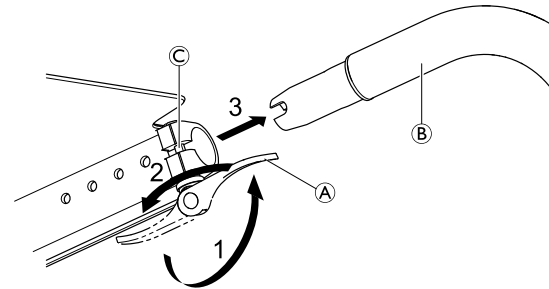

 The manual opening and closing of the quick release lever requires increased activation force of approx. 60 to 100 Newton.

 If an active brake is fitted, this must be activated when removing or attaching the quick release frame.

Removing the quick release frame



1. Open the quick release levers **A** on both sides by pushing them forwards.
2. Turn back the open quick release levers by 180° on both sides so the frame tubes **B** are unlocked by the pins **C**.
3. Pull out both frame tubes **B** to the front.

IMPORTANT!

When the quick release frame is removed, the foot brace fitted inside can warp if force is applied to its frame tubes. Likewise the quick release couplings can be damaged.

- After removal, carefully store the quick release frame.
- Ensure that the quick release couplings are protected when stored.

Attaching the quick release frame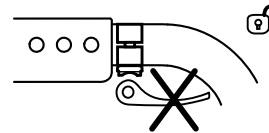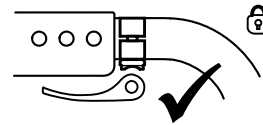

WARNING!

Danger of severe injury

Closing the quick release lever in the direction of movement does not guarantee mechanical locking. The front frame section can come apart.

- Make sure that the quick release lever is turned by 180° against the direction of movement when closed.

1. To attach the quick release frame, follow the instructions for removal in reverse order.


3.4 Push handles

IMPORTANT!

- Always check the push handles before using the wheelchair, as to whether the hand grips are secure, cannot be turned and cannot be pulled off.

Foldable push handles (optional)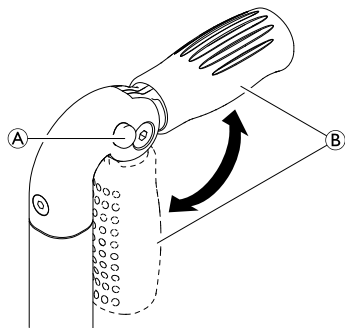

1. Fully depress button **A** and fold up respectively fold down the push handle **B** until it audibly engages.


CAUTION!

If not correctly engaged, the push handle could accidentally fold down while pushing the wheelchair.

- Make sure that the push handle is correctly engaged.


Height adjustable, angled push handles (optional)

The infinitely height adjustable push handles allow any assistant to adjust the handles to a comfortable level.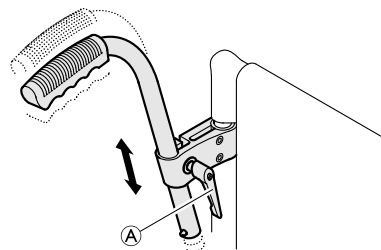

1. To set the height of the push handles, loosen the bolt **A**, push the handle to the desired position and tighten the bolt again.

3.5 Side rest, stepless height adjustable, non-locking

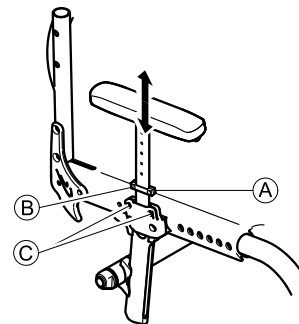WARNING!
Risk of injury

The side rests are not locked and can be easily pulled out upwards.

- Do not lift or transfer the wheelchair using the side rests.
- Do not use the side rests for transportation when carrying the wheelchair up- or downstairs.

Removing

1. Pull the side rest by the arm cushion out of the holder.
2. To adjust how easy or difficult it is to pull the side rest out of the holder, change the degree to which the screws © are tightened.


Fitting

1. Push the side rest into the holder.

Adjusting the height

1. Loosen the screw ⑥ of the retaining plate ⑤.
2. Move the side rest upwards or downwards until you reach the desired height.
3. Tighten the screw ⑥.
4. Perform the setting on both sides.

3.6 Simple armrest, height-adjustable, swivelling, non-locking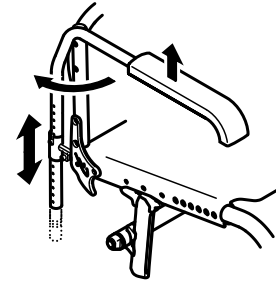

WARNING!
Risk of injury

The armrests are not locked and can be easily pulled out upwards.

- Do not lift or transfer the wheelchair using the armrests.
- Do not use the armrests for transportation when carrying the wheelchair up- or downstairs.

Removing

1. Raise the armrest and pull it upwards out of the holder.

Fitting

1. Push the armrest into the holder.

Adjusting the height

1. Pull the armrest out of the holder.
2. Loosen the screw on the tube of the armrest and screw it back in at the desired height.


3. Push the armrest back into the holder.
4. Perform the setting on both sides.

Swivelling

1. Raise the armrest a little and swivel it outwards.

3.7 Mudguard

In order to prevent dirt from the wheels spraying upwards, a removable mudguard can be fitted. The position of the mudguard can be adjusted. The adjustment must be carried out by the specialist dealer.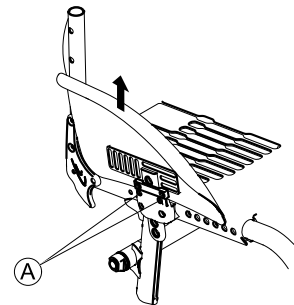

Removing


1. Pull the mudguard upwards out of the holder.
2. To adjust how easy or difficult it is to pull the mudguard out of the holder, change the degree to which the screws **A** are tightened.

Fitting

1. Push the mudguard into the holder.

3.8 Clothes-guard

A clothes-guard can be fitted to protect your clothes. Before folding the back of the wheelchair, the clothes-guard must first be folded inwards against the backrest.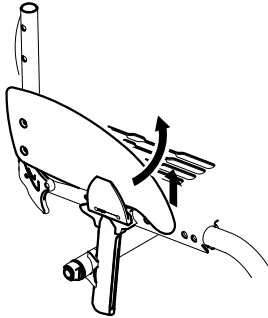


Folding in

1. Raise the clothes-guard at the front and swivel it inwards in front of the backrest.

3.9 Footrest

The height of the footrest can be adjusted. The adjustment must be carried out by the specialist dealer.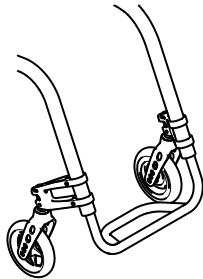


3.10 Backrest

Backrest cover

You can adjust the tension of the backrest cover as required.

Adjusting the backrest cover

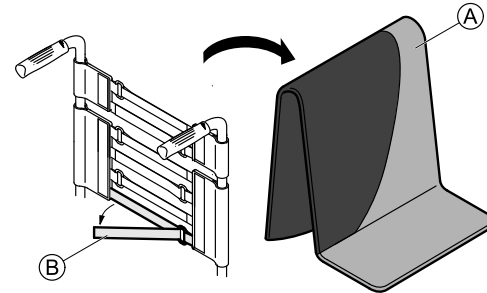1. Remove the backrest cushion **(A)**.
2. Loosen the Velcro® tapes **(B)** on the rear of the backrest cover by simply pulling on them.
3. Tighten or loosen the tapes as required then reattach them.

WARNING! **Risk of tipping**

If the bands are set very loose, the tipping point of your wheelchair is changed for the worse.
– Ensure that the bands are set correctly.

Backrest height

The height of the backrest can be adjusted. The adjustment must be carried out by the specialist dealer.

Backrest angle

The angle of the backrest can be adjusted. The adjustment must be carried out by the specialist dealer.

Backrest bar

WARNING!

Risk of accidents

The backrest bar is not suited for lifting or carrying the wheelchair with the user sitting in it as it can break.

- The backrest bar may not be used to lift or carry the wheelchair while the user is sitting in it.

3.11 Antitipper

An antitipper prevents the wheelchair from tipping backwards.

WARNING!

Risk of overturning

Antitippers that are incorrectly set or no longer working can lead to overturning.

- Always check that the antitipper is working before using the wheelchair and have it set or readjusted by a specialist dealer when required.


WARNING!

Risk of overturning

On uneven or soft ground, the antitipper can sink into potholes or directly into the ground, thereby curtailing or eliminating its safety function.

- Only use the antitipper when travelling on even and firm ground.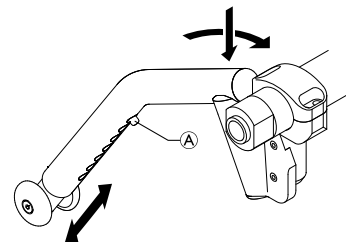

Activating the antitipper

1. Push the antitipper completely downwards against the spring and swivel it backwards through 90° until it engages.

WARNING!

Risk of tipping

An activated antitipper can catch when negotiating a step or an edge.

- Always deactivate the antitipper before driving over a step or kerb.

Deactivating the antitipper

1. Push the antitipper completely downwards against the spring and swivel it inwards through 90° until it engages.

Setting the height


1. Press the spring pin **A** on the antitipper and push the inner section to the position required. Allow the spring pin to engage in the nearest hole.

If further adjustments of height and/or position are necessary, refer to your specialized dealer.

3.12 Vario-Ax

Your wheelchair can be fitted with a Vario-Ax. The Vario-Ax makes it possible to change the wheel camber.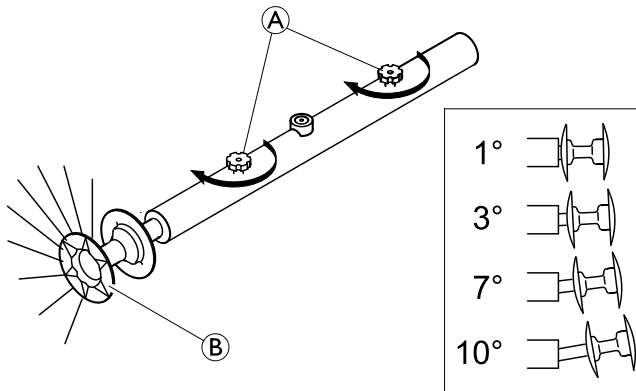

Adjusting wheel camber of the Vario-Ax

! **IMPORTANT!**
– Do not overtighten the knurled screw (A) as this can damage its mechanism.

! **IMPORTANT!**
– Changing the wheel camber can impair or negate the function of the parking brakes.

1. Loosen the knurled screw (A) on the Vario-Ax.
2. Pull the wheel by the hub (B) until the desired angle is set.
3. A gentle engagement will indicate when it has reached one of the four possible positions.

4. Retighten the knurled screw (A).
5. Perform the same setting on both sides.

3.13 Tires

The ideal pressure depends on the tire type:

The table below is an indication. In case the tire differs from the list below, check the side of the tire, the maximum pressure is often listed there.

Tire	Max. pressure		
Light wheel	7 bar	700 kPa	101 psi
Profile tire	7 bar	700 kPa	101 psi
Schwalbe® Marathon Plus tire	10 bar	1000 kPa	145 psi
Schwalbe® One	10 bar	1000 kPa	145 psi
Schwalbe® Rightrun, blue	10 bar	1000 kPa	145 psi
Solid tire, profile, grey	-	-	-
Solid tire (KIK type), black	-	-	-
Solid, light wheel	-	-	-

The compatibility of the tires listed above depends on the configuration and/or model of your wheelchair.

3.14 Seat cushion

A suitable cushion is needed to provide an even pressure distribution on the seat.

Use a seat cushion with anti-slip underlay or Velcro® (hook) fastening tapes to avoid slipping of the seat cushion. A Velcro® (loop) fastening tape is pre-attached to the seat cover.

3.15 Seat position

Tipping stability

The seat position and thus the tipping stability of your wheelchair can be altered stepless. The adjustment must be carried out by the specialist dealer.

Seat height front

The seat height front can be stepless adjusted. The adjustment must be carried out by a specialist dealer.


Seat height rear

The seat height rear can be stepless adjusted. The adjustment must be carried out by a specialist dealer.

4 ACCESSORIES

4.1 Pelvic belt

Your wheelchair can be fitted with a pelvic belt which will prevent you from sliding forward in the wheelchair or from falling out of the wheelchair.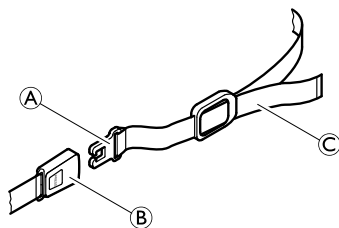

WARNING!

Risk of serious injuries through strangulation.

- Ensure that you are sitting fully back in the seat and that the pelvis is as upright and symmetrical as possible - not forward on one side or tilted back.
- Position the lap belt so that the hip bones can be felt above the belt.
- Adjust the length using the buckles so that there is just sufficient room for your hand to slide between your body and the belt. It is recommended that the clasp is kept in a central position, i.e. make adjustments to each side.
- Make sure that the belt cannot slide and the clasp does not dissolve. Refit the pelvic belt if necessary, ® specialist dealer.
- Check and possibly change these adjustments each time the belt is used.

Opening the pelvic belt

1. Push in the PRESS button and pull the catch ① out of the buckle ②. The pelvic belt opens.

Closing the pelvic belt

1. Push the catch ① into the buckle ②.
2. Pull the belt ③ until you are held securely in the wheelchair.


4.2 Transit wheels

If your wheelchair is too wide for certain applications, e.g. trains/aeroplanes, narrow passageways, narrow doors, the transit wheels can be used.

WARNING!

– When using transit wheels, the parking brakes no longer have any effect and you cannot control your wheelchair via the handrims.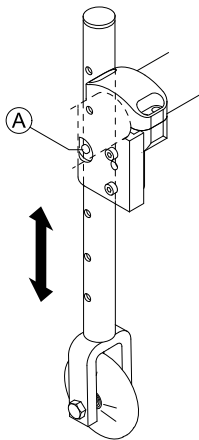

Switching to transit wheels

1. Lower the transit wheels into the operating position. To do this, press the pin **A** inwards until you can pull the tube down.
2. Press the second pin in.
3. Pull the tube further downwards until the pin engages into the slot.

4. Move the wheelchair to a stable object (door frame, heavy table), which you can use for support and hold onto it firmly with one hand.
5. Tilt the wheelchair until the rear wheel on the opposite side rises off the floor.

Prior to removing the rear wheels, the distance between the lowered transit wheels and the floor should not be more than 2 cm.

6. Remove the rear wheel,
 @ 7.3 Removing and fitting the rear wheels, page 34.
7. Repeat the process on the other side.

Switching from transit wheels to rear wheels

1. Move the wheelchair to a stable object, which you can use for support and hold onto it firmly with one hand.
2. Tilt the wheelchair until the transit wheel on the opposite side rises slightly off the floor.
3. Fit the rear wheel back onto the removable axle,
 @ 7.3 Removing and fitting the rear wheels, page 34.
4. Bring the transit wheels back into the inactive position.
5. Repeat the process on the other side.

4.3 Luggage carrier

Your wheelchair can be fitted with a luggage carrier. When not in use, the luggage carrier can be folded up.


WARNING!

Risk of tipping

Weight on the carrier changes the tipping behaviour of the wheelchair. The wheelchair can tip forwards more easily when the luggage carrier is loaded.

- Use the luggage carrier only for transporting small and lightweight luggage.
- Avoid driving the wheelchair too fast when carrying luggage.
- Avoid driving ramps and slopes when carrying luggage.
- Do not stand on the luggage carrier.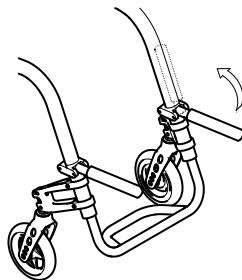

4.4 Air pump

The air pump is fitted with a universal valve connection.

1. Remove the dust cap from the valve connection.
2. Push the valve connection onto the open valve on the wheel and pump the wheel up.

4.5 Passive illumination

You can attach reflectors to the rear wheels.

5 SETUP

5.1 Safety information

CAUTION!

Risk of injury

- Before using the wheelchair, check its general condition and its main functions, 8.2 Maintenance Schedule, page 36.

Your specialist dealer will supply your wheelchair ready for use. Your dealer will explain the main functions and ensure that the wheelchair meets your needs and requirements.

Adjustments of the axle position and the front wheel supporters must be carried out by a specialist dealer.

6 USING THE WHEELCHAIR

6.1 Safety information

**WARNING!****Risk of accidents**

Uneven tire pressure can have a huge effect on handling.

- Check the tire pressure before each journey.

**WARNING!****Risk of falling out of the wheelchair**

When using too small front wheels the wheelchair could get stuck at curbs or in floor grooves.

- Make sure that the front wheels are suitable for the surface you're driving on.

**CAUTION!****Risk of crushing**

There may be a very small gap between the rear wheel and the mudguard with the risk that you could trap your fingers.

- Ensure that you always propel your wheelchair using the handrims only.

**CAUTION!****Risk of crushing**

There may be a very small gap between the rear wheel and the parking brake with the risk that you could trap your fingers.

- Ensure that you always propel your wheelchair using the handrims only.

Theft and metal detection systems

In seldom cases the materials used in the wheelchair may activate theft and metal detection systems.

6.2 Braking during use

Whilst you are moving, you brake by transferring force to the handrim with your hands.

WARNING! **Risk of overturning**

If you apply the parking brakes while you are moving, the direction of movement can become uncontrollable and the wheelchair may stop suddenly, which can lead to a collision or to you falling out.

- Never apply the parking brakes while you are moving.

WARNING! **Risk of falling out of the wheelchair**

If the wheelchair is rapidly decelerated by an assistant pulling at the push handles, the user may fall out of the wheelchair.

- Always apply the pelvic belt if present.
- Make sure your assistant has been trained individually in transferring occupied wheelchairs.

CAUTION! **Risk of burning your hands**

If you brake for a long time, a lot of frictional heat is produced at the handrims (especially MaxGrepp and Supergripp).

- Wear suitable gloves.

1. Hold the handrims and press evenly with both hands until the wheelchair stops.

6.3 Getting in and out of the wheelchair

WARNING! **Risk of overturning**

There is a high risk of overturning during the transfer.

- Only get in and out without assistance if you are physically able to do so.

WARNING! **Risk of overturning**

The wheelchair could tip forwards if you stand on the footrest.

- Never stand on the footrest when getting in and out.

CAUTION!

If you release or damage the brakes the wheelchair could roll away out of control.

- Do not support yourself on the brakes when getting in and out.

IMPORTANT!

The mudguards and armrests could become damaged.

- Never sit on the mudguards or armrests when getting in and out.

1. Apply the parking brakes.
2. Place your feet on the ground.
3. Hold the wheelchair and, if necessary, also hold a fixed object in the vicinity.
4. Move slowly to chair.

6.4 Driving and steering the wheelchair

You drive and steer the wheelchair using the handrims. Before driving without an assistant you must find your wheelchair's tipping point.

WARNING!

Risk of tipping

The wheelchair can tip backwards if it is not fitted with an antitipper. When finding the tipping point, an assistant must stand immediately behind the wheelchair to catch it if it tips over.

- To prevent tipping, install an antitipper device.

WARNING!

Risk of tipping

The wheelchair can tip forwards.

- When setting up your wheelchair, test its behaviour in terms of tipping forward and adjust your driving style accordingly.


CAUTION!

A heavy load hanging on the backrest can affect the wheelchair's centre of gravity.

- Change your driving style accordingly.

Finding the tipping point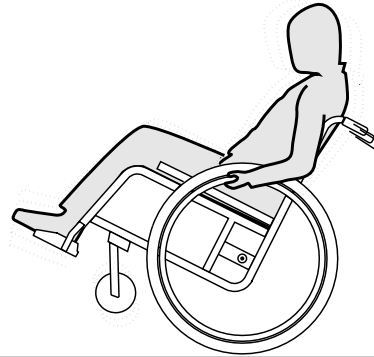

1. Release the brake.
2. Roll forwards a short distance, grasp both handrims firmly and pull backwards with a slight kick.
3. The shift in weight and steering in opposite directions with the handrims will enable you to identify the tipping point.

6.5 Negotiating steps and kerbs

WARNING! **Risk of overturning**

When negotiating steps you could lose your balance and tip the wheelchair over.

- Always approach steps and kerbs slowly and carefully.
- Do not go up or down steps that are higher than 25 cm.

CAUTION!

An activated antitipper prevents the wheelchair from tipping backwards.

- Deactivate the antitipper before going up or down steps or kerbs.

With an assistant

Going down a step

1. Move the wheelchair right up to the kerb and hold the handrims.
2. The assistant should hold both push handles, place one foot on the tipper aid (if installed) and tilt the wheelchair backwards so that the front wheels lift off the ground.
3. The assistant should then hold the wheelchair in this position, push it carefully down the step and then tilt it forwards until the front wheels are back on the ground.

Going up a step


WARNING! **Risk of serious injuries**

Going up steps and kerbs frequently can result in earlier than expected fatigue break of the wheelchair backrest. The user might fall out of the wheelchair.

– Always use a tipper aid when going up steps or kerbs.

1. Move the wheelchair backwards until the rear wheels touch the kerb.
2. The assistant should tilt the wheelchair using both push handles so that the front wheels lift off the ground, then pull the rear wheels over the kerb until the front wheels can be placed back on the ground.

Without an assistant

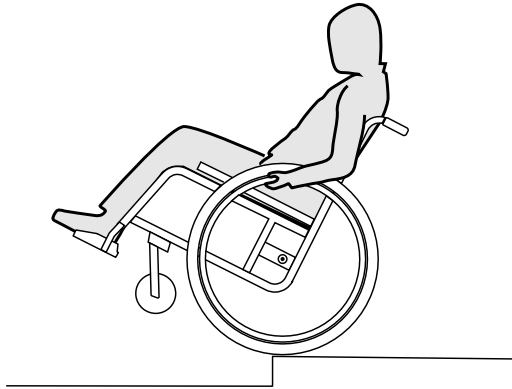WARNING! **Risk of tipping**

When going down a step without an assistant you could tip over backwards if you cannot control your wheelchair.

- First learn how to go down a step with an assistant.
- Learn how to balance on the rear wheels, 6.4 Driving and steering the wheelchair, page 27.

Going down a step

1. Move the wheelchair right to the kerb, lift the front wheels and keep the wheelchair balanced.
2. Now slowly roll both rear wheels over the kerb. While doing this, hold the handrims firmly with both hands until the front wheels are back on the ground.

6.6 Going up and down stairs

WARNING! **Risk of overturning**

When negotiating stairs you could lose your balance and overturn your wheelchair.

- Always use two assistants when negotiating stairs with more than one step.

1. You can go up and down stairs by taking them one step at a time, as described above. The first assistant should stand behind the wheelchair holding the push handles. The second assistant should hold a solid part of the front frame to steady the wheelchair from the front.

6.7 Negotiating ramps and slopes

WARNING! **Risk due to wheelchair being out of control**

When negotiating slopes or gradients your wheelchair could tip backwards, forwards or sideways.

- Always have an assistant behind the wheelchair when approaching long slopes.
- Avoid lateral slopes.
- Avoid slopes of more than 7°.
- Avoid jerking when changing direction on a slope.


CAUTION!

Your wheelchair could run away even on slightly sloping ground if you do not control it using the handrims.

- Use the parking brakes if your wheelchair is stationary on sloping ground.

Going up slopes


To go up a slope, you must create some momentum, keep up the momentum and control the direction at the same time.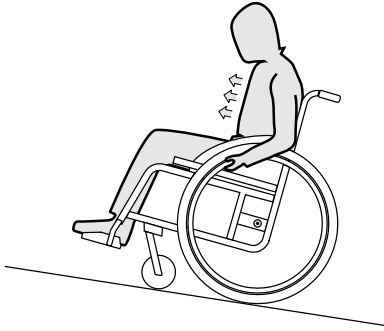

1. Bend your upper body forwards and propel the wheelchair with quick, powerful strokes on both handrims.

Going down slopes

When going down slopes, it is important to control your direction and particularly your speed.

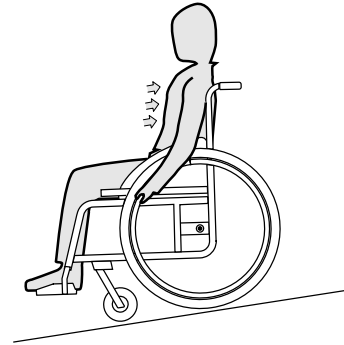1. Lean back and carefully allow the handrims to run through your hands. You should be able to stop the wheelchair at any time by gripping the handrims.

CAUTION!

Risk of burning your hands.

If you brake for a long time, a lot of frictional heat is produced at the handrims (especially MaxGripp and Supergripp).

– Wear suitable gloves.

6.8 Stability and balance when seated

Some everyday activities and actions require you to lean forwards, sideways or backwards out of the wheelchair. This has a major effect on the wheelchair's stability. To keep your balance at all times, proceed as follows:

Leaning forwards


WARNING!

Risk of falling out

If you lean forwards out of the wheelchair you could fall out of it.

- Never bend too far forwards and do not shift forwards in your seat to reach an object.
- Do not bend forwards between your knees to pick up something off the floor.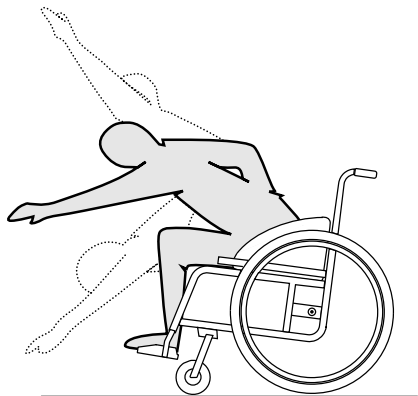

1. Point the front wheels forwards. (To do this, move your wheelchair forwards slightly then back again.)
2. Apply both parking brakes.
3. When leaning forwards your upper body must remain over the front wheels.

Reaching backwards


WARNING!

Risk of falling out

If you lean too far backwards you could tip your wheelchair over.

- Do not lean out over the backrest.
- Use an antitipper device.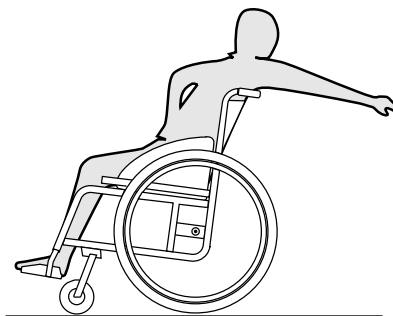

1. Point the front wheels forwards. (To do this, move your wheelchair forwards slightly then back again.)
2. Do not apply the parking brakes.
3. When reaching backwards do not reach so far that you have to change your sitting position.

7 TRANSPORT

7.1 Safety information

WARNING!

Risk of injury if wheelchair is not secured during transport

Your wheelchair is not designed for passenger transport in vehicles. You could suffer serious injury in an accident.

- During transport, sit in a correctly secured seat, not in the wheelchair.

WARNING!

Risk of injury if the wheelchair is not properly secured

In the event of an accident, braking manoeuvre, etc. you may suffer serious injuries from flying wheelchair parts.

- Always remove the rear wheels when transporting the wheelchair.
- Firmly secure all wheelchair components in the means of transport to prevent them from coming loose during the journey.


IMPORTANT!

Excessive wear and abrasion could affect the strength of load-bearing parts.

- Do not pull your wheelchair across abrasive surfaces without the wheels fitted (e.g. pulling the frame over tarmac).

7.2 Folding and unfolding the wheelchair

The wheelchair has a rigid frame. Nevertheless, the backrest can be folded forwards and fixed there.

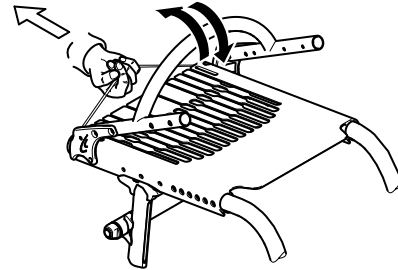Folding the wheelchair

1. Remove the seat cushion, if present.
2. Fold the clothes-guard, if present, inwards against the backrest on both sides.
3. Pull the cord on the back of the wheelchair and fold the backrest forward until it engages.
4. The wheelchair can now be lifted using the backrest brace.

Unfolding the wheelchair

IMPORTANT!

- When unfolding the wheelchair, do not pull the backrest brace without previously releasing the backrest using the cord.
- Check that the backrest is completely engaged on the frame, before you use the wheelchair again.

1. Pull the cord on the backrest of the wheelchair.
2. Pull the backrest backwards using the backrest brace, until the backrest tube engages on both sides of the frame.
3. Fold the clothes-guard back into position on both sides.

7.3 Removing and fitting the rear wheels


WARNING!

Risk of overturning

If the removable axle of a rear wheel is not fully engaged, the wheel can become loose during use. This can lead to overturning.

- Always ensure that the removable axles are fully engaged whenever you fit a wheel.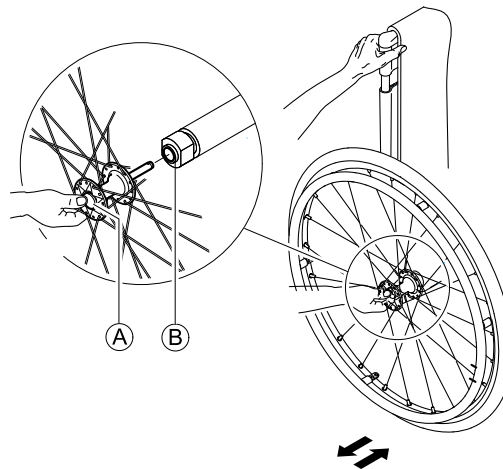

Removing the rear wheels

1. Release the brakes.
2. With one hand, hold the wheelchair upright.
3. With the other, hold the wheel through the external spoke rim around the wheel hub.
4. Using your thumb, press the removable axle button (A). Keep it pressed and pull the wheel out of the adapter sleeve (B).

Fitting the rear wheels

1. Release the brakes.
2. With one hand, hold the wheelchair upright.
3. With the other, hold the wheel through the external spoke rim around the wheel hub.
4. Using your thumb, press the removable axle button and hold it down.
5. Push the axle into the adapter sleeve ® up to the stop.
6. Release the removable axle button and make sure that the wheel is secure.

8 MAINTENANCE

8.1 Safety information

WARNING!

Some materials deteriorate naturally over time. This could result in damage to wheelchair components.

- Your wheelchair should be checked by a specialist dealer at least once a year or if it has not been used for a long period.

Your *küschall*® specialist dealer can provide help in carrying out regular maintenance. To find an authorised specialist dealer near you, contact the *küschall*® distributor for your country (end of this document).

8.2 Maintenance Schedule

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person.

	weekly	monthly	annually
Check the tire pressure	x		
Check that rear wheels are seated correctly	x		
Checking the adjustment of the backrest joint	x		
Check the pelvic belt	x		

	weekly	monthly	annually
Visual check		x	
Clean the front wheels		x	
Check bolts		x	
Check spokes		x	
Check parking brakes		x	
Have wheelchair checked by a specialist dealer			x

Check the tire pressure

1. Check the tire pressure, 3.13 Tires, page 19.
2. Inflate the tires to the required pressure.
3. Check the tire tread at the same time.
4. If necessary, change the tires.

Check that rear wheels are seated correctly

1. Pull on the rear wheel to check that the removable axle is seated correctly. The wheel should not come off.
2. If the rear wheels are not engaged properly, remove any dirt or deposits. If the problem persists, have the removable axles re-fitted by a specialist dealer.

Check the adjustment of the backrest joint

1. Sit into the wheelchair and lean back on the backrest. The backrest must be correctly engaged.
2. Pull the release cord. The pins must be free-moving on both sides. Otherwise, the backrest joint will have to be re-fitted by a specialist dealer.

Check the pelvic belt

1. Make sure the pelvic belt is adjusted properly.

IMPORTANT!

- Loose pelvic belts must be adjusted by a specialist dealer.
- Damaged pelvic belts must be replaced by a specialist dealer.

Visual check

1. Examine your wheelchair for loose parts, cracks or other defects.
2. If you find anything, have your wheelchair checked immediately by a specialist dealer.

Clean the front wheels

1. Check that the front wheels turn freely.
2. Remove any dirt or hair from the front wheel bearings.

Check the bolts

Bolts can work loose through constant use.

1. Check that the bolts are tight (on the footrest, seat cover, sides, backrest, frame, seat module).
2. Tighten any loose bolts with the suitable torque. Therefore refer to the Service Manual küschall Advance®, available on the internet at www.kueschall.com.

IMPORTANT!

- Safety nuts and bolts lose their effectiveness as a result of repeated loosening and tightening.
- Safety nuts and bolts must be replaced by a specialist dealer.

Check the spoke tension

The spokes should not be loose or distorted.

1. Loose spokes must be tightened by a specialist dealer.
2. Broken spokes must be replaced by a specialist dealer.

Check the parking brakes

1. Check that the parking brakes are positioned correctly. The brake is set correctly if the brake shoe depresses the tire by a few millimeters when the brake is applied.
2. If you find that the setting is not correct, have the brakes correctly set by a specialist dealer.

IMPORTANT!

- The parking brakes must be reset after replacing the rear wheels or changing their position.

Checking after a heavy collision or blow

IMPORTANT!

- The wheelchair can sustain visibly undetectable damage as a result of a heavy collision or hard blow.
- It is essential to have your wheelchair checked by a specialist dealer after a heavy collision or hard blow.

Repairing or changing an inner tube

1. Remove the rear wheel and release any air from the inner tube.
2. Lift one tire wall away from the rim using a bicycle tire lever. Do not use sharp objects such as a screwdriver which could damage the inner tube.
3. Pull the inner tube out of the tire.

4. Repair the inner tube using a bicycle repair kit or, if necessary, replace the tube.
5. Inflate the tube slightly until it becomes round.
6. Insert the valve into the valve hole on the rim and place the tube inside the tire (the tube should lie right round the tire with no creases).
7. Lift the tire wall over the edge of the rim. Start close to the valve and use a bicycle tire lever. When doing this, check all the way round to ensure that the inner tube is not trapped between the tire and the rim.
8. Inflate the tire to the maximum operating pressure. Check that no air is escaping from the tire.

Spare parts

All spare parts may be obtained from a küschall® specialist dealer. A current list of spare parts is available on the internet at www.kueschall.com.

8.3 Cleaning

Your wheelchair will serve you well for many years if you clean it regularly.

IMPORTANT!

Sand and seawater can damage the bearings and steel parts can rust if the surface is damaged.

- Only expose the wheelchair to sand and seawater for short periods and clean it after every trip to the beach.

IMPORTANT!

- Do not use coarse abrasives, aggressive cleaning products or high-pressure cleaners.

1. Clean the cushions and metal parts with a soft, damp cloth.
2. Dry the wheelchair carefully with a cloth after using it in the rain or in the shower.
3. If the wheelchair is dirty, wipe off the dirt as soon as possible with a damp cloth and dry it carefully.

8.4 Disinfection

The wheelchair may be disinfected by spraying or wiping with tested, approved disinfectants.

The Robert Koch Institute has a list of currently approved disinfectants at www.rki.de.

9 TROUBLESHOOTING

9.1 Safety information

Faults may arise as a result of daily use, adjustments or changing demands on the wheelchair. The table below shows how to identify and repair faults.

**CAUTION!**

- Contact your specialist dealer immediately if you notice a fault with your wheelchair, e.g. a significant change in handling.

**IMPORTANT!**

- Some of the actions listed must be carried out by an authorised specialist dealer. These are indicated. We recommend that *all* adjustments are carried out by a specialist dealer.

9.2 Identifying and repairing faults

Fault	Possible cause	Action
The wheelchair does not travel in a straight line	Incorrect tire pressure on one rear wheel	Correct tire pressure, ® 3.13 Tires, page 19
	One or more spokes broken	Replace faulty spoke(s), ® specialist dealer
	Spokes tightened unevenly	Tighten loose spokes, ® specialist dealer
	Front wheel bearings are dirty or damaged	Clean or replace the bearings, ® specialist dealer
	Front wheel supporter is not aligned correctly	Correct position and alignment of front wheel supporter, ® specialist dealer
The wheelchair tips backwards too easily	Rear wheels are mounted too far forwards	Mount the rear wheels further back, ® specialist dealer
	Back angle too large	Reduce the backrest angle, ® specialist dealer
	Seat angle too large	Use longer vertical struts, ® specialist dealer
The brakes are gripping poorly or asymmetrically	Incorrect tire pressure in one or both rear tires	Correct tire pressure, ® 3.13 Tires, page 19
	Brake setting incorrect	Correct the brake setting, ® specialist dealer
Rolling resistance is very high	Tire pressure in rear tires is too low	Correct tire pressure, ® 3.13 Tires, page 19
	Rear wheels not parallel	Ensure the rear wheels are parallel, ® specialist dealer
The front wheels wobble when moving fast	Too little tension in front wheel bearing block	Tighten the nut on the bearing block axle slightly, ® specialist dealer
	Front wheel has worn smooth	Change front wheel, ® specialist dealer
The front wheel is stiff or stuck	Bearings are dirty or faulty	Clean or replace the bearings, ® specialist dealer
Carbon fibres are sticking out	Overstress or damage of material	Smooth with grinding paper (grit 80) or contact specialist dealer
Front frame wobbles (option, quick release frame)	Loss of clamping force due to loose quick release clamps	Readjust quick release device, ® specialist dealer

10 AFTER USE

10.1 Storage

**IMPORTANT!**

Risk of damage to the product

- Do not store the product near heat sources.
- Never store other items on top of the wheelchair.
- Store the wheelchair indoors in a dry environment.
- Refer to section 11.2 Environmental conditions, page 43.

After long-term storage (more than four months) the wheelchair must be inspected in accordance to chapter 8 Maintenance, page 36.

10.2 Re-use

The wheelchair is suitable for re-use. The following actions should be taken:

- Cleaning and disinfection, 8.3 Cleaning, page 38 and 8.4 Disinfection, page 38 in this manual.
- Inspection, 8.2 Maintenance Schedule, page 36 in this manual.
- The wheelchair should be adapted for the new user in accordance with the service documentation (available from the *küschall*® distributor in your country).

10.3 Disposal

Be environmentally aware and dispose of your wheelchair properly. Disposal is subject to national and local regulations.


For correct disposal, contact your specialist dealer or ask your town or district council about local waste management companies.

11 TECHNICAL DATA

11.1 Dimensions and weight

All dimension an weight specifications refer to a seat width of 400 mm, seat depth of 400 mm in the lightest configuration of the wheelchair. Dimension an weight may alter according to different configurations.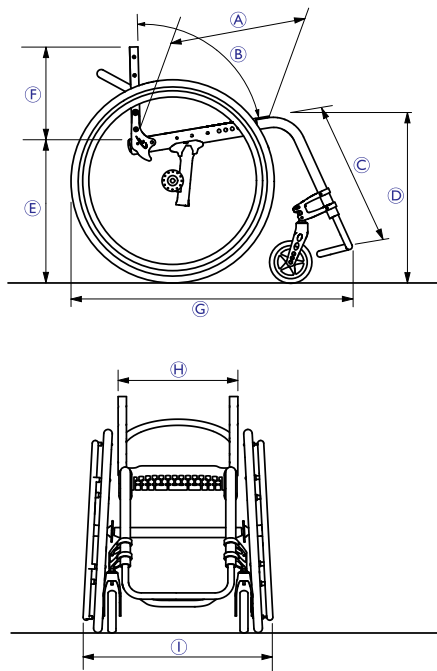

Ⓐ	Seat depth	360 – 520 mm, in increments of 20 mm
Ⓑ	Backrest angle	78°/82°/86°/90°/94°
Ⓒ	Knee-to-heel length	220 – 520 mm, stepless adjustable
Ⓓ	Seat height front	450 – 550 mm, stepless adjustable
Ⓔ	Seat height rear	390 – 490 mm, stepless adjustable
Ⓕ	Backrest height	270 – 480 mm, in increments of 15 mm
Ⓖ	Total length	75°: approx. 830 mm 90°: approx. 760 mm
Ⓗ	Seat width	340 – 460 mm, in increments of 20 mm
Ⓘ	Total width	Seat width + 170 mm
	Total Weight	approx. 8,8 kg
	Transport weight (without rear wheels)	approx. 6,2 kg
	Maximum user weight	125 kg
	Reversing width	920 – 1230 mm
	Center of gravity	80 – 140 mm, stepless adjustable
	Max. safe slope	7°

11.2 Environmental conditions

Do not expose the wheelchair to temperatures below -20 °C or above 40 °C.

11.3 Materials

The components used to manufacture küschall® wheelchairs consist of following materials:

Frame tubes / Backrest tubes	Aluminium / Titanium ¹⁾ / Carbon ¹⁾
Seat cover / Backrest cover	PA / PE / PVC
Push handle	Steel / Aluminium / PVC
Clothes guard / Mudguard	Plastic or Carbon ¹⁾
Supporting parts / Attachments	Steel / Aluminium / Titanium ¹⁾
Screws and bolts	Steel
Castors	Aluminium

¹⁾ Not for all küschall® wheelchair models Carbon and/or Titanium options are available.

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Invacare Asia Ltd.