USER MANUAL

VICAIR®

HYGIENE

relief by air

EN	USER MANUAL	3
NL	GEBRUIKERSHANDLEIDING	6
DE	GEBRAUCHSANWEISUNG	10
FR	MANUEL DE L'UTILISATEUR	14
DA	BRUGERVEJLEDNING	18
SV	BRUKSANVISNING	21
NO	BRUKERMANUAL	24
FI	KÄYTTÖOHJE	27
ES	MANUAL DE USUARIO	30
PT	MANUAL DO UTILIZADOR	33
IT	MANUALE UTENTE	37
GR	ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ	41
HR	KORISNIČKI PRIRUČNIK	45
SL	UPORABNIŠKI PRIROČNIK	48
PL	INSTRUKCJA UŻYTKOWNIKA	51
ZH	用户手册	55
JP	ユーザーマニュアル	58

Thank you for choosing a Vicair® product. Please read and understand this user manual before use. Save this manual for future reference.

For more information or to (re)order manuals, please see vicair.com or contact your country's distributor.

Go to vicair.com for distributor information.

INTRODUCTION

The Vicair Hygiene is a seat cushion for use in the Aquatec Ocean Ergo (VIP) shower and commode chair. It can be used in other shower and commode chairs if all precautions and safety instructions in this user manual are followed. It provides skin protection for wheelchair users with a risk of developing pressure injuries. It is also an addition to a pressure injury treatment plan (stage I-IV).


INDICATIONS

The cushion is used to provide skin protection and comfort.

CONTRAINDICATIONS

When used as intended, there are no known contraindications.

MAIN COMPONENTS

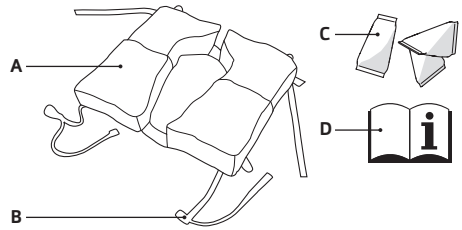- A. Vicair Hygiene
- B. Fixation straps
- C. SmartCells™
- D. User manual

GENERAL PRODUCT SPECIFICATIONS

Size:	One size
Used materials:	All latex free
Top:	Fluid-resistant polyurethane coated polyester fabric
Bottom:	Knitted polyester mesh fabric
Filling:	Air-filled SmartCells with low friction coating

PRECAUTIONS AND SAFETY INSTRUCTIONS

The Vicair Hygiene is a seat cushion designed for use in the Aquatec Ocean Ergo (VIP) shower and commode chair. It can be used in other shower and commode chairs if all precautions and safety instructions are followed. The use of this product in any other way is considered to be misuse. Vicair BV cannot be held responsible for damage caused by misuse, improper use or negligence.

Warning! Vicair recommends to install the Vicair Hygiene under the direct supervision of your therapist or Assistive Technology Professional.

Warning! Check your skin regularly for redness, especially during first use. If redness occurs and does not disappear after 15 minutes, do not continue to use the cushion. Consult your therapist or Assistive Technology Professional.

Warning! Ensure the dimensions of the Vicair Hygiene match the dimensions of the shower and commode chair.

Warning! The cushion must be used as directed in this manual. Do not place obstructions or non-compatible accessories between the individual and the cushion. Placement of any items between the individual and the cushion and/or using the cushion on top of, or in conjunction with, any other products or materials: 1) may reduce or eliminate the benefits of the cushion, increasing the risk to skin or to other soft tissue, and 2) may cause the individual to become unstable and vulnerable to falling.

Warning! To ensure stability, use the fixation straps to secure the Vicair Hygiene to the shower and commode chair.

Warning! If the Vicair Hygiene feels unstable or can be moved by hand do not use until properly secured.

Warning! The user should always keep their weight centrally over the toilet seat. Leaning excessively to either side could cause instability and injury. If recommended by your health care professional the user should be supervised or assisted when using this product.

Warning! Check your cushion every three months for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the product. Consult your Assistive Technology Professional.

Caution! Do not use or leave the product close to a naked flame or a high temperature heat source.

Caution! Do not dry clean, steam clean or autoclave the cushion.

Caution! Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

Caution! Do not place the topside of the cushion against the glass of the washer door.

Caution! Excessive washing may influence the aging of this product.

Notice! Filling grade may slightly deviate from default (see product label).

INSTALLATION

These steps correspond to the installation images 1 – 5 on the Quick Installation Guide:

- Place the cushion on the (Aquatec Ocean Ergo (VIP)) shower and commode chair.
- Check if the cushion fits the dimensions of the chair and make sure the opening in the cushion corresponds to the opening in your seat.
- Put the fixation straps (4x) around the canes of your chair, close the buckles and tighten the straps.

USE

- This Vicair product is ready for use.
- Dry your cushion after each use by tumble drying your cushion on air only setting. Or remove excessive water by spin drying your cushion and hang dry your cushion.

Conditions for use and storage

User weight: Minimum: 15 kg / 33 lbs,
maximum: 250 kg / 551 lbs

Temperature: Minimum: -20 °C / 4 °F,
maximum: 55 °C / 131 °F

Use at high altitude **Notice!** Vicair recommends not to use the cushion above 2400 m (8000 ft) above sea level or below 50m (164 ft) below sea level.

Re-use - Transferring the product to another user

The product has to be thoroughly cleaned and disinfected before another person is allowed to use it. See chapter 'Maintenance and cleaning' for instructions.

Notice! Make sure the product is correctly fastened to the other chair.

Expected lifetime

The expected lifetime of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective lifetime can vary according to frequency and intensity of use.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulation. Ensure the product is cleaned prior to disposal to avoid any risk of contamination.

MAINTENANCE AND CLEANING

Cleaning the Vicair Hygiene

Your Vicair Hygiene cushion can be machine washed at 60 °C/140 °F using an anti-crease cycle and mild antibacterial detergent. You do not need to remove the SmartCells before putting the cushion into the washing machine. Do not load the machine with more than half of its maximum load weight. Spin speed should be reduced. When using a machine with agitator, make sure that the cushion can move freely within the drum. Do not use any chlorine bleach! Dry your cushion after each use and after washing by tumble drying your cushion on air only setting. Or remove excessive water by spin drying your cushion and hang dry your cushion.

Disinfecting your Vicair Hygiene

Warning! After washing spray your cushion with a disinfectant. Bend compartments outwards to make sure the spray gets into the gaps.

REPORTING OF SERIOUS INCIDENTS

Serious incidents from use of Vicair products should be reported to the manufacturer and the relevant competent authority.

WARRANTY

Vicair offers a limited warranty against defects in workmanship and materials for a period of 24 months after the original date of purchasing. For general warranty conditions see our website www.vicair.com.

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Vicair B.V.
ProPharma Group MIS Limited