# Invacare® XPO<sub>2</sub>™

XPO100, XPO100B

en	<b>Portable Oxygen Concentrator</b> User Manual .....	3
el	<b>Φορητός συμπυκνωτής οξυγόνου</b> Εγχειρίδιο χρήσης .....	39
es	<b>Concentrador de oxígeno portátil</b> Manual del usuario .....	85
it	<b>Concentratore di ossigeno portatile</b> Manuale d'uso .....	125
nl	<b>Draagbare zuurstofconcentrator</b> Gebruiksaanwijzing .....	165
pt	<b>Concentrador de oxigénio portátil</b> Manual de utilização .....	203

This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, read this manual and save for future reference.

**Yes, you can.®**

© 2014 Invacare® Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Pine-sol and Lestoil are trademarks of The Clorox Company.

Dawn, Fairy and Dreft are trademarks of The Proctor and Gamble Company.

---

# Contents

---

This manual **MUST** be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

<b>I General</b> .....	<b>4</b>
1.1 Symbols .....	4
1.2 Intended Use .....	5
1.2.1 Description .....	5
1.3 Features .....	6
1.3.1 User Interface .....	6
1.3.2 Indicators, Warnings, and Alarms .....	7
1.3.3 Filters .....	7
1.3.4 Power Supplies .....	8
<b>2 Safety</b> .....	<b>9</b>
2.1 Typical Product Parameters .....	9
2.2 Regulatory Listing .....	11
2.3 General Guidelines .....	12
2.4 Radio Frequency Interference .....	16
2.5 Traveling with the XPO <sub>2</sub> .....	17
<b>3 Accessories</b> .....	<b>20</b>
3.1 Optional Accessories .....	20
<b>4 Setup</b> .....	<b>21</b>
4.1 Unpacking .....	21
4.2 Inspection .....	21
4.3 Location .....	21
<b>5 Usage</b> .....	<b>22</b>
5.1 Getting Started .....	22
5.2 Connecting/Positioning the Nasal Cannula .....	22
5.3 Using the XPO <sub>2</sub> Portable Concentrator .....	23

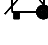
5.4 Adjusting the Pulse Flow Setting .....	24
5.5 Checking Charge Level of Internal Battery .....	25
5.6 Charging the Internal Battery .....	25
5.7 Charging the Supplemental Battery .....	26
5.8 Charging the Internal and Supplemental Battery Pack Simultaneously .....	27
5.9 Battery Life and Management .....	28
<b>6 Maintenance</b> .....	<b>29</b>
6.1 Cleaning the Cabinet .....	29
6.2 Cleaning the Air Intake Filter .....	29
6.3 Cleaning the Carrying Bag .....	30
6.4 Maintenance Checklist .....	31
<b>7 After Use</b> .....	<b>32</b>
7.1 Storage .....	32
7.2 Disposal .....	32
<b>8 Troubleshooting</b> .....	<b>33</b>
8.1 Troubleshooting .....	33
<b>9 Warranty</b> .....	<b>38</b>
9.1 Limited Warranty - Europe and Australia .....	38

## I General

### I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the following table for definitions of the signal words.

	<p><b>DANGER!</b> Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.</p> <p><b>WARNING!</b> Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.</p> <p><b>CAUTION!</b> Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.</p>
	Gives useful tips, recommendations and information for efficient, trouble-free use.
	<b>IMPORTANT</b> indicates a hazardous situation that could result in damage to property if it is not avoided.
	This product complies with Directive 93/42/EEC concerning medical devices.  The launch date of this product is stated in the CE declaration of conformity.
	Consult the user manual.
	Direct Current

	Type BF equipment
	DO NOT smoke
	Class II equipment
	Power On/Off
	DO NOT dispose of in household waste
	Recycle
	DO NOT use oil or grease
	Keep dry
<b>IPX1</b>	Protected against dripping water
<b>R<sub>x</sub></b>	Restricted to sale by or on order of a medical professional

## 1.2 Intended Use

The Invacare portable oxygen concentrator is to be used by patients with respiratory disorders who require supplemental oxygen. The device is not intended to sustain or support life.

### **DANGER!** **RISK OF INJURY OR DEATH**

This product is to be used as an oxygen supplement and is not intended to be life-supporting or life-sustaining. ONLY use this product if the patient is capable of spontaneous breath, able to inhale and exhale without the use of a machine.

- DO NOT use in parallel or series with other oxygen concentrators or oxygen therapy devices.

### **WARNING!** **RISK OF INJURY OR DAMAGE**

Use of this product outside of the intended use and product parameters has not been tested and may lead to product damage, loss of product function, or personal injury.

- DO NOT use this product in any way other than described in the product parameters and intended use sections of this manual.

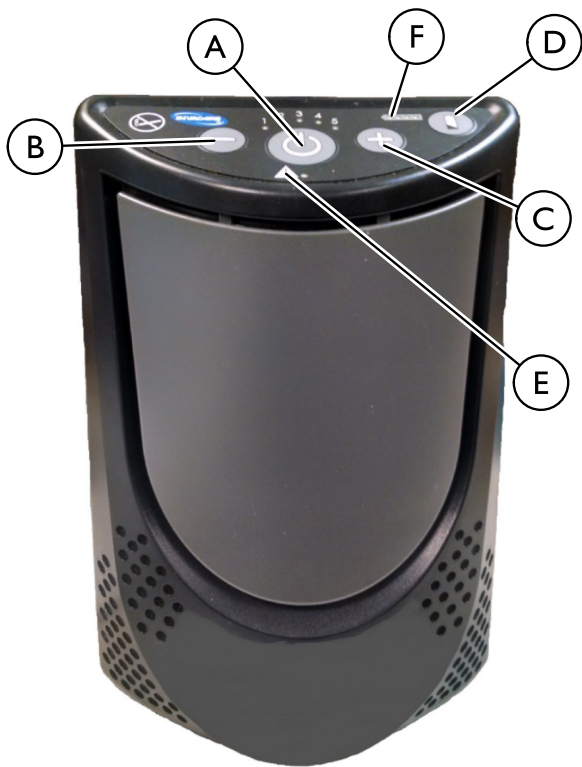
## 1.2.1 Description

The oxygen concentration level of the output gas ranges from 87% to 95.6%. The oxygen is delivered to the patient through the use of a nasal cannula. When the demand for oxygen is detected, the oxygen is delivered through pulsed flow with pulse flow settings of 1 through 5.

The Invacare portable concentrator is capable of operation by the patient in a home environment, in an institutional environment or in a vehicle or other mobile environment. Device standard power options include an AC to DC switching power supply operating from AC power outlet (120 VAC/ 60 Hertz or 230VAC/50 Hertz nominal), a DC to DC switching power supply operating from accessory outlets typically found in a mobile vehicle type environment (12 VDC nominal) and a supplemental rechargeable battery.

## 1.3 Features

### 1.3.1 User Interface

Ⓐ	On/off button
Decrease Ⓑ Increase Ⓒ	Flow Selection buttons  The XPO <sub>2</sub> concentrator can be adjusted to your prescribed flow setting by pressing the flow selection (+ or -). The selected flow setting is illuminated by a blue indicator light.
Ⓓ	Battery Status button  Press once to determine the charge level of the internal battery.
Ⓔ	Alarm Indicator  Indicates abnormal conditions and warns the user. Refer to 8.1 Troubleshooting, page 33 for more details.
Ⓕ	Battery Gauge Display  Indicates the level of battery charge remaining.

### 1.3.2 Indicators, Warnings, and Alarms

<u>Start-Up Indicator</u>	The XPO <sub>2</sub> concentrator will turn on all lights briefly and check for stuck buttons upon power up. It will then output a brief audible beep when first turned on to indicate the unit is operational.
<u>Breath Detect Indicator</u>	The XPO <sub>2</sub> concentrator will deliver a measured pulse of oxygen each time it detects an inhalation. The blue flow setting indicator light will blink each time an inhalation is detected and the XPO <sub>2</sub> concentrator outputs a pulse of oxygen.
<u>Internal Battery Charging Indicator</u>	<p>When the concentrator is not turned on, but the AC, or DC, charger is connected to the concentrator and providing power, the Battery Gauge Display will sequentially illuminate each of the four segments. Periodically the sequential illumination will be interrupted by the actual battery charge display.</p> <p>If the concentrator is fully charged, all four segments will be illuminated continuously until the concentrator is turned on, or removed from charger power. If no Battery Gauge Display segments are illuminated, the concentrator is not charging. The lack of charging could be due to either a loss of power, loose charger connections or the internal battery is not at the proper temperature for charging. Allow the unit to warm, or cool, to recommended charging temperature range (see 2.1 Typical Product Parameters, page 9), check for loose connections and/or try a different power outlet.</p>
<u>Audible Alarms</u>	To temporarily silence the audible alarms, quickly press the On/Off button. Except for the system Too Hot/Too Cold alarms, this feature will reset itself, every 30 seconds, each time a new alarm is triggered, or if the unit is turned Off. When performed during the system Too Hot/Too Cold alarms, the audible alarm will be silenced until a new alarm is triggered, or the concentrator is turned Off.

For all alarms and warnings, refer to 8.1 Troubleshooting, page 33.

To clean/replace the air intake filter, refer to 6.2 Cleaning the Air Intake Filter, page 29.

### 1.3.3 Filters

Air enters the XPO<sub>2</sub> concentrator through an air intake filter located under the cover on the front of the unit. This filter prevents hair and other large particles in the air from entering the unit. Before you operate the XPO<sub>2</sub> concentrator, make sure this filter is clean, dry and properly positioned.

### 1.3.4 Power Supplies

#### **WARNING!**

#### **Risk of Injury or Damage**

To avoid injury or damage which will void warranty:

- Use only Invacare specified power supplies.

The XPO<sub>2</sub> concentrator can be supplied power from a factory installed internal battery, from an AC power adapter connected to the wall outlet, from a DC power adapter connected to a DC outlet like that found in an automobile, or from a supplemental battery pack.

**Internal Battery:** A rechargeable, internal battery is located inside the XPO<sub>2</sub> concentrator. When fully charged, it supplies power for up to 3.5 hours. An audible alarm sounds when the battery power is getting low. Refer to 1.3.2 Indicators, Warnings, and Alarms, page 7.

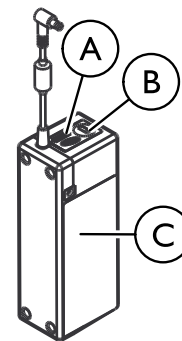
**AC Power Adapter:** An AC power adapter allows the XPO<sub>2</sub> concentrator to be connected to a 100-230 volt 50/60 hertz outlet. Use of the AC power adapter will allow the XPO<sub>2</sub> concentrator to be operated and simultaneously recharge the internal battery. This AC power adapter can also be used to recharge the supplemental battery pack. Refer to 5.6 Charging the Internal Battery, page 25, 5.7 Charging the Supplemental Battery, page 26 and 5.8 Charging the Internal and Supplemental Battery Pack Simultaneously, page 27.

**DC Power Adapter:** A DC power adapter allows the XPO<sub>2</sub> concentrator to be connected to an automobile's (boat, motor home, etc....) 12-volt DC outlet. Use of the DC power adapter will allow the XPO<sub>2</sub> concentrator to be operated and simultaneously recharge the internal battery. This DC power adapter can also be used to recharge the supplemental battery pack. Refer to 5.6 Charging the Internal Battery, page 25, 5.7 Charging the Supplemental Battery, page 26 and 5.8 Charging the Internal and Supplemental Battery Pack Simultaneously, page 27.

**Supplemental Battery Pack:** The XPO<sub>2</sub> concentrator can be powered by a supplemental battery pack ©. This battery pack can be attached to the XPO<sub>2</sub> carrying bag and connected to the power input connector of the unit. The supplemental battery pack will provide up to 3.5 hours of additional run time. The supplemental battery pack has its own battery status button Ⓑ and battery gauge Ⓐ to identify the level of battery charge.

The supplemental battery cannot be used to charge the internal battery.

## 2 Safety

### 2.1 Typical Product Parameters

Electrical Requirements:	AC Power Supply: 120VAC 60 Hz and 230VAC 50 Hz
	DC Power Supply: 11VDC, 6.5A
Rated Current Input:	3.8 A at 18 VDC
Sound Level at setting 2:	≤ 42.7dBA weighted
Altitude:	Up to 10,000 ft (3046 m) above sea level
Oxygen Concentration: *Based on an atmospheric pressure of 14.7 psi (101 kPa) at 70°F (21°C)	87% to 95.6%, after initial warm-up period (approximately 5 minutes)
Conserver Trigger Sensitivity: Factory set — no adjustment Pressure activated	≤ 0.20 cmH <sub>2</sub> O pressure drop All settings — using 7 ft (2.1 m ) cannula
Conserver Trigger Delay: Factory set — no adjustment	50 mSec Nominal value — using 7 ft (2.1 m ) cannula
Conserver Breath Rate Capacity:	Up to, and including, 35 BPM without reduction of bolus minute volume
Conserver Bolus Delivery: Fixed minute volume	Setting 1: 300 cc Setting 2: 460 cc Setting 3: 620 cc Setting 4: 740 cc Setting 5: 840 cc
Dimensions:	10" high x 7" wide x 4" deep (25.4 cm high x 17.8 cm wide x 10.2 cm wide)

Weight:	6.40 lbs (2.90 kg); 1.20 lbs (0.54 kg) for supplemental battery
Battery Duration: *Times are approximate (Hr:Min)	Setting 1 = 3:40 hrs
	Setting 2 = 2:40 hrs
	Setting 3 = 2:00 hrs
	Setting 4 = 1:40 hrs
	Setting 5 = 1:20 hrs
Battery Recharge Time:  Recharge time increases if battery is charging while unit is running.	4 hours
Humidity	Operating Humidity: 15% to 60% non condensing
	Storage Humidity: up to 95% non condensing
Temperature Range: (All power sources)	Operating temperature: 41°F to 95°F (5°C to 35°C)
	Storage temperature: -2°F to 140°F (-20°C to 60°C)
Extended Temperature Range: (Using AC or DC adapters)	Operating Temperature: 95°F to 104° (35°C to 40°C) Continuous use - all settings

<p>Extended Temperature Range (Using Internal Battery) *Times are approximate (Hr:Min)</p>	<p>Operating Temperature: 95°F to 104°F (35°C to 40°C)</p> <p>Settings 1 = 3:40 hrs Setting 2 = 2:40 hrs Setting 3 = 2:00 hrs Setting 4 = 1:45 hr Setting 5 = 1:10 hr</p>
--	---

## 2.2 Regulatory Listing

<p>ETL certified complying with:</p>	<p>EN 55011:2009</p>
	<p>CISPR 11: 2009</p>
	<p>IEC 60601-1; 2nd ed.</p>
	<p>IEC 60601-1-2; 3rd ed. 2007</p>
	<p>IEC 61000-3-2:2006</p>
	<p>IEC 61000-3-3:2008</p>
	<p>UL 60601-1, 1st ed.</p>
	<p>CSA 601.1 M90</p>
	<p>EN 60601-1-2: 2007</p>
	<p>EN 61000-3-2: 2006</p>
	<p>EN 61000-3-3: 2008</p>
	<p>93/42/EEC Class IIa CENELEC EN 60601-1:1990</p>

## 2.3 General Guidelines

In order to ensure the safe installation, assembly and operation of the XPO<sub>2</sub> concentrator these instructions MUST be followed.

### **DANGER!**

#### **Risk Of Death, Injury, Or Damage**

Improper use of the product may cause death, injury or damage. This section contains important information for the safe operation and use of this product.

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment.
- If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment.
- Check ALL external components and carton for damage. In case of damage, or if the product is not working correctly, contact a technician or Invacare for repair.
- THE INFORMATION IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

### **DANGER!**

#### **Risk Of Death, Injury, Or Damage From Fire**

Textiles, oil or petroleum substances, grease, greasy substances and other combustibles are easily ignited and burn with great intensity in oxygen enriched air and when in contact with oxygen under pressure. To avoid fire, death, injury or damage:

- DO NOT SMOKE while using this device.
- DO NOT use near OPEN FLAME or IGNITION SOURCES.
- DO NOT use any lubricants on concentrator unless recommended by Invacare.
- NO SMOKING signs should be prominently displayed.
- Avoid creation of any spark near oxygen equipment. This includes sparks from static electricity created by any type of friction.
- Keep all matches, lighted cigarettes or other sources of ignition out of the room in which this concentrator is located and away from where oxygen is being delivered.
- Keep the oxygen tubing, cord, and concentrator out from under such items as blankets, bed coverings, chair cushions, clothing, and away from heated or hot surfaces including space heaters, stoves, and similar electrical appliances.

**DANGER!****Risk of Death, Injury, from Electric Shock**

To reduce the risk of burns, electrocution, death or injury to persons:

- DO NOT disassemble. Refer servicing to qualified service personnel. There are no user serviceable parts.
- Avoid using while bathing. If continuous usage is required by the physician's prescription, the concentrator must be located in another room at least 7ft (2.5m) from the bath.
- DO NOT come in contact with the concentrator while wet.
- DO NOT place or store concentrator where it can drop into water or other liquid.
- DO NOT reach for concentrator that has fallen into water. Unplug IMMEDIATELY.
- DO NOT use frayed or damaged AC adapter cords.

**WARNING!****Risk Of Injury Or Damage**

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

**CAUTION!**

Federal (statutory) law restricts this device to sale by or on the order of a medical practitioner licensed by a governmental agency where he/she practices.

- ONLY a licensed medical practitioner may order the purchase or use of this device.

**DANGER!****Risk of Injury or Death**

To avoid choking or ingestion of chemicals from airway contamination:

- DO NOT use the concentrator in the presence of pollutants, smoke, fumes, flammable anesthetics, cleaning agents, or chemical vapors.

**WARNING!****Risk of Injury or Death**

To prevent injury or death from product misuse:

- Closely supervise when this concentrator is used by or near children or physically-challenged individuals.
- Monitor patients using this device who are unable to hear or see alarms or communicate discomfort.

**WARNING!****Risk of Injury or Death**

To reduce the risk of injury or death from illness:

- Replace the nasal cannula on a regular basis. Check with your equipment provider or physician to determine how often the cannula should be replaced.
- DO NOT share cannulas between patients.

**WARNING!**

**Risk of Injury**

It is important to plan ahead for travel and other situations in which you may not have access to additional oxygen or power supplies. To prevent injury from oxygen deprivation:

- Carry supplemental batteries with you.
- Stow the concentrator properly during travel. For proper storage instructions when travelling refer to the travel guide.
- Obtain and carry a licensed physician's signed statement on a standard form (p/n 1154266), the doctor's letterhead, or an airline-approved form during travel.

**WARNING!**

**Risk of Injury or Damage**

To prevent injury or damage from cord misuse:

- DO NOT move or relocate concentrator by pulling on the cord.
- DO NOT use extension cords with AC power adapters provided.
- Properly store and position electrical cords and/or tubing to prevent a tripping hazard.

**WARNING!**

**Risk of Injury or Damage**

To prevent injury or damage from misuse:

- NEVER leave concentrator unattended when plugged in.
- Make sure concentrator is off when not in use.
- Outdoor use MUST be conducted with internal or supplemental battery power only.

**WARNING!**

**Risk of Injury or Damage**

To prevent injury or damage during use:

- If you feel ill or uncomfortable, or if the concentrator does not signal an oxygen pulse and you are unable to hear and/or feel the oxygen pulse, consult your equipment provider and/or your physician IMMEDIATELY.
- For optimum performance, Invacare recommends that each concentrator be on and running for a minimum of 30 minutes. Shorter periods of operation may reduce maximum product life.
- For maximum oxygen output, the concentrator should be used in an upright position.
- The XPO<sub>2</sub> concentrator cannot be used in conjunction with PAP, Bi-Level, mechanical ventilator or other such devices.

**WARNING!****Risk of Injury**

A change in altitude may affect total oxygen available to you. To prevent oxygen deprivation:

- Consult your physician before traveling to higher or lower altitudes to determine if your flow settings should be changed.

**WARNING!****Risk of Injury or Damage**

As a safety feature, this appliance may have a polarized plug (one blade is wider than the other). To avoid injury or damage from electrical shock:

- This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician.
- DO NOT alter the plug in any way.
- DO NOT use if the cord or plug are damaged. Call a qualified electrician for repair.

**WARNING!****Risk of Injury or Damage**

Invacare oxygen concentrators are specifically designed to minimize routine preventive maintenance. To prevent injury or damage:

- Only professionals of the healthcare field or persons fully conversant with this process such as factory trained personnel should perform preventive maintenance or performance adjustments on the oxygen concentrator, except for tasks described in this manual.
- Users should contact your dealer or Invacare for service.

**CAUTION!****Risk of Minor Injury or Discomfort**

During the initial start up time (app. 30 minutes) oxygen output is not within specifications listed in Typical Product Parameters.

- Concentrator may be used during start up time.

**CAUTION!****Risk of Damage**

To prevent damage from liquid ingress:

- If the XPO<sub>2</sub> concentrator is not working properly, if it has been dropped or damaged, or dropped into water, call equipment provider/qualified technician for examination and repair.
- NEVER drop or insert any object or liquid into any opening.
- Invacare recommends the XPO<sub>2</sub> concentrator not be used in the rain.
- The XPO<sub>2</sub> concentrator is not designed to be used with a humidifier. Use of this device with a humidifier may impair performance and/or damage the equipment.

**CAUTION!****Risk of Damage**

To prevent damage from temperature variations:

- DO NOT operate in temperatures below 41° F (5°C) or above 104° F (40° C) for extended periods of time.
- When your automobile is turned Off, disconnect the car accessory power supply and remove the XPO<sub>2</sub> concentrator from the automobile. NEVER allow the XPO<sub>2</sub> concentrator to be stored in a very hot or cold automobile or in other similar, high or low, temperature environments. Refer to 2.1 Typical Product Parameters, page 9 .

## 2.4 Radio Frequency Interference

This equipment has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2. These limits are designed to provide a reasonable protection against electromagnetic interference in a typical medical installation.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standards. To determine if the emissions from the XPO<sub>2</sub> concentrator are causing the interference, turn the XPO<sub>2</sub> concentrator Off. If the interference with the other device(s) stops, then the XPO<sub>2</sub> concentrator is causing the interference. In such rare cases, interference may be reduced or corrected by one of the following measures:

- Reposition, relocate, or increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

DO NOT connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices.

## 2.5 Traveling with the XPO<sub>2</sub>

### Before Traveling

1. Contact your travel carrier and/or tour guide about your travel plans and provide information about your intent to use a portable oxygen concentrator (POC) during your travels. The carrier will let you know if you will be permitted to use your XPO<sub>2</sub> concentrator during your travel and if there are any restrictions for use.

 If you are not permitted to use your XPO<sub>2</sub> concentrator on the airline of choice and still want to take it with you:

- Carry it and any supplemental batteries on board with you.
- Ensure the unit is turned off.
- Stow it properly for the flight.
- DO NOT place the XPO<sub>2</sub> concentrator or any supplemental battery modules in checked baggage.

2. Contact the airline to see if they have seats with a power outlet that you can use to power the XPO<sub>2</sub> concentrator during the flight. If a power outlet is not available, be sure to bring enough supplemental batteries to last 150% of the flight time.

 There may be unexpected delays beyond scheduled flight times. Carry your power adapters with you so you can recharge batteries before and after the flight.

3. Ensure you have a licensed physician's signed statement on a standard form (p/n 1154266), the doctor's letterhead or an airline-approved form that contains the following information:

- Your need for oxygen during taxi, takeoff, flight times, landing, or the entire time you are traveling.
- The flowrate(s) that you will use during your trip.
- Your understanding, or that of your travel companion, of all the cautions, warnings, and alarms of the XPO<sub>2</sub> concentrator and what to do if there is an alarm situation with the device.
- Your personal information such as name, date of birth, and the model name or number of the portable concentrator.

 **You MUST have this form with you at all times during your travel.**

4. Charge the internal battery of the XPO<sub>2</sub> concentrator and any supplemental batteries fully before you depart, thus extending the operating time of the XPO<sub>2</sub> concentrator during travel. Refer to Recharging the Battery in the owner's manual.
5. Ensure the unit is free of all grease, oil, or other petroleum products and that the unit is in good working order, is free of damage, and the air filter is clean. Refer to Maintenance in the owner's manual.
6. Contact your oxygen supplier to make arrangements if you need to have backup oxygen at your destination.

## Arriving at the Airport

1. Allow the security agent to inspect your XPO<sub>2</sub> concentrator, even if you are using it, when going through the security checkpoint.

 They are required to allow you to travel through the checkpoint with your POC, but they MUST inspect it for security reasons.

2. Use your AC power adapter while you are in the airport, if possible. This will keep the batteries at full charge and give you the most battery powered operating time on the aircraft.

## Boarding the Plane

 You are not permitted to sit in an exit row if you plan to use your XPO<sub>2</sub> concentrator at any time during the flight.

1. If you plan on using the XPO<sub>2</sub> concentrator the entire time you are the plane, verify that your XPO<sub>2</sub> concentrator does not block access to an emergency exit or aisle way. If it does, request a seat change.
2. Show the pilot or flight crew your signed physician's statement regarding your use of the XPO<sub>2</sub> concentrator, if asked.
3. In order for the flight crew to ensure the safety of you and all of the other passengers on the flight, allow them to inspect the XPO<sub>2</sub> concentrator to confirm that it is FAA approved for use on the flight.
4. If you are not planning to use your XPO<sub>2</sub> concentrator during taxi, takeoff, or landing, you MUST store it in an approved storage area so it does not block the row or aisle access.

 Contact the flight crew about location of approved storage areas.

## On the Plane

 You MUST store the supplemental battery either securely connected to the XPO<sub>2</sub> concentrator or in your carry-on baggage. The supplemental battery MUST be protected from damage and shorting out the external power connector.

1. You may use your XPO<sub>2</sub> concentrator when moving about the cabin, ONLY after the pilot turns off the "Fasten Seat Belt" sign.
2. Turn off the unit if the XPO<sub>2</sub> concentrator alarms during the flight, unless the alarm resets itself. If the alarm does not reset, the XPO<sub>2</sub> concentrator MUST be turned off and stored in an approved storage area.
3. If an AC power outlet is available on the flight, perform the following:
  - a. Connect the power adapter to the XPO<sub>2</sub> concentrator.
  - b. Plug in the power adapter into the airline power outlet.
  - c. Turn the unit on.

 If you have any trouble making the connections to the plane's power outlet, contact the flight crew for assistance.

## After the Flight

1. Ensure you have enough power to run your XPO<sub>2</sub> concentrator while leaving the airport. Recharge the batteries, if necessary. Refer to Recharging the Battery in the owner's manual.
2. Contact your medical gas supplier to obtain your backup oxygen supply, if necessary.

## Traveling by Boat

Contact the cruise line and inform them you are traveling with a POC. There should be no restrictions on your use of the XPO<sub>2</sub> concentrator during your travel but it is wise to check with them

ahead of time. Power should be available for use of your AC power adapter during your travels, but check first.

### **Traveling by Train**

Contact the train authorities at least twelve hours ahead of arrival that you are going to use your POC. There should be no restrictions on your use of the XPO<sub>2</sub> concentrator, but power on the train may not be available for your use during travel times. Ensure there is enough battery life for your trip before you leave.

### **Traveling by Bus**

Contact the bus line about using a POC. There should be no restrictions on your use of the XPO<sub>2</sub> concentrator during your trip, but power may not be available for your use during travel times. Ensure there is enough battery life for your trip before you leave.

## 3 Accessories

---

### 3.1 Optional Accessories

The following optional accessories and replacement parts (listed below) are also available:

- Supplemental battery (provides up to additional 2.40 (Hr:Min) on Setting 2 of extended battery life), model number XPO110
- Carrying Bag, model number XPO150
- Wheeled cart with handle, model number XPO120
- Air intake filter, part number 1157692
- DC power adapter, model number XPO140
- Accessory bag, model number XPO160 (Unit model XPO100B only)
- AC power adapter, power cord style:
  - North America, Japan — model number XPO130
  - Europe — model number XPO132
  - Australia, New Zealand — model number XPO134
  - United Kingdom, Hong Kong, Vietnam — model number XPO136

## 4 Setup

### 4.1 Unpacking

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier, or your local dealer.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton. The Invacare XPO<sub>2</sub> concentrator's packaging contains the following items (as shown below). If any parts are missing, please contact your equipment provider.
  - XPO<sub>2</sub> with carrying bag
  - User manual
  - AC power adapter
  - DC power adapter
  - Supplemental battery (EU and AZ model XPO100B only)
  - Accessory bag (EU and AZ model XPO100B only)

Retain all containers and packing materials for storage or return shipment.

### 4.2 Inspection

Inspect/examine exterior of the oxygen concentrator and accessories for damage. Inspect all components.

### 4.3 Location

#### **WARNING!**

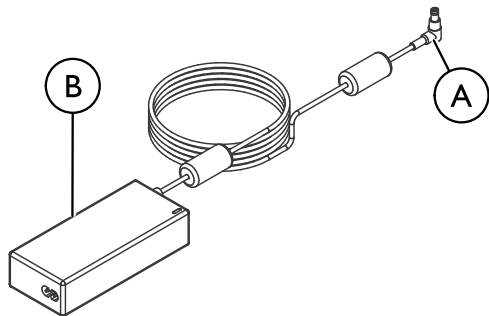
#### **Risk of Injury or Damage**

To avoid injury or damage from airborne pollutants and/or fumes and for optimal performance:

- Locate and position the concentrator in a well ventilated space so that the air intake and the air exhausts are not obstructed.
- NEVER block the air openings of the concentrator or place it on a soft surface, such as a bed or couch, where the air opening may be blocked.
- Keep the openings free from lint, hair and similar foreign items.
- Keep concentrator at least 12 inches (30,5 cm) away from walls, draperies and furniture.
- Avoid use in presence of pollutants, smoke or fumes, flammable anesthetics, cleaning agents or chemical vapors.
- Place concentrator in a well ventilated area to avoid airborne pollutants and/or fumes.
- DO NOT use in a closet.

## 5 Usage

### 5.1 Getting Started

Using the XPO<sub>2</sub> concentrator for the first time requires the internal battery to be charged. To charge the battery, perform the following steps:

1. Connect the AC power adapter (A) to the input power connector located on the side of the XPO<sub>2</sub> concentrator.
2. Connect the other end of the power adapter (B) into the power source.

#### **!** IMPORTANT

- A concentrator will not turn on unless the battery is charged.
  - Connect the AC power to the XPO<sub>2</sub> concentrator and allow battery to fully charge prior to first use.

### 5.2 Connecting/Positioning the Nasal Cannula

#### **DANGER!**

##### **Risk of Injury or Death**

The cannula can cause tripping, falling, or other injury if improperly positioned and secured. To avoid injury or death:

- The cannula **MUST** be routed and secured properly.
- **DO NOT** position the cannula around the neck. Ensure the patient can move freely while wearing the cannula.
- Avoid positioning cannulas across areas of high foot traffic (i.e. aisles, doorways, hallways, etc.)

#### **WARNING!**

##### **Risk of Injury or Death**

To avoid choking and/or strangulation from tubing entanglement:

- Keep children and pets away from nasal cannula and tubing.
- Close supervision is necessary when the nasal cannula is used by or near children and/or disabled persons.

#### **CAUTION!**

##### **Risk of Minor Injury**

To ensure proper breath detection and oxygen delivery:

- **DO NOT** use tubing length exceeding 25 ft (7.6 m).
- Use crush-proof oxygen tubing.

Invacare recommends using the XPO<sub>2</sub> concentrator with a 4 ft (1.2 m) cannula. The tubing must be connected to the XPO<sub>2</sub> concentrator oxygen outlet as shown below.

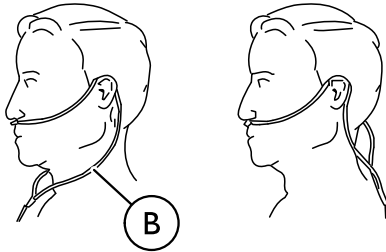
Replace the nasal cannula on a regular basis. Check with your equipment provider or physician to determine how often the cannula should be replaced.

DO NOT share cannulas between patients.

**DETAIL "A"**

**DETAIL "B"**

1. Connect the nasal cannula to the XPO<sub>2</sub> concentrator oxygen outlet port (A) (Detail "A").
2. Place the cannula (B) over your ears and position the prongs in your nose as instructed by your health care provider or cannula manufacturer (Detail "B").

### 5.3 Using the XPO<sub>2</sub> Portable Concentrator

1. Turn the XPO<sub>2</sub> concentrator On by pressing the On/Off button.
2. Breathe normally through your nose. Breathing through your mouth may result in less effective oxygen therapy. The XPO<sub>2</sub> concentrator will put out a measured pulse of oxygen each time it detects inhalation. The blue flow setting indicator light will blink each time an inhalation is detected and the XPO<sub>2</sub> concentrator outputs a pulse of oxygen.
3. Turn the XPO<sub>2</sub> concentrator Off by pressing and holding the power On/Off button for approximately two seconds.

The time required for the XPO<sub>2</sub> concentrator to reach maximum oxygen concentration after being turned On is approximately 30 minutes.

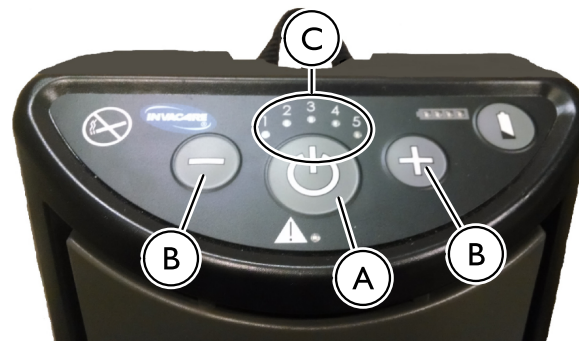
### Recommendations for Optimal Performance

Operating Temperature:	41° F to 95° F (5° C to 35° C)
Relative Humidity:	15% to 60%
Transport/Storage Temperature:	-2° F to 140° F (-20°C to 60°C) Allow unit to warm, or cool, to operating temperature range before using.
Electrical:	No extension cords.

Altitude:	Up to 10,000 ft (3046 m) above sea level.
Tubing and Cannula:	4 ft (1.2 m) cannula with a maximum 25 ft (7.6 m) of crush resistant tubing (DO NOT pinch).
Environment:	Smoke, pollutant and fume free. No confined spaces (example: no closets).
Time of Operation:	Up to 24 hours per day when connected to AC or DC power adapter.

### 5.4 Adjusting the Pulse Flow Setting

FLOW SETTING	AVERAGE PULSE VOLUME (AT 20 BREATHS PER MINUTE)
1	16 ml
2	24 ml
3	31 ml
4	37 ml
5	42 ml

1. Turn On the XPO<sub>2</sub> concentrator by pressing and holding the power On/Off button (A) for approximately one second. Each time the XPO<sub>2</sub> concentrator is turned On, a brief alarm will sound. The XPO<sub>2</sub> concentrator pulse flow setting at power-up will be the same flow setting selected the last time the unit was turned Off.
2. The XPO<sub>2</sub> concentrator has five pulse flow settings. Press the flow selection (+ or -) button (B) until your prescribed flow setting (1 to 5) is illuminated by a blue indicator light (C).

**WARNING!**  
**Risk of Injury**

It is very important to select the prescribed level of oxygen flow.

- DO NOT increase or decrease the flow unless a change has been prescribed by your physician or therapist.

## 5.5 Checking Charge Level of Internal Battery

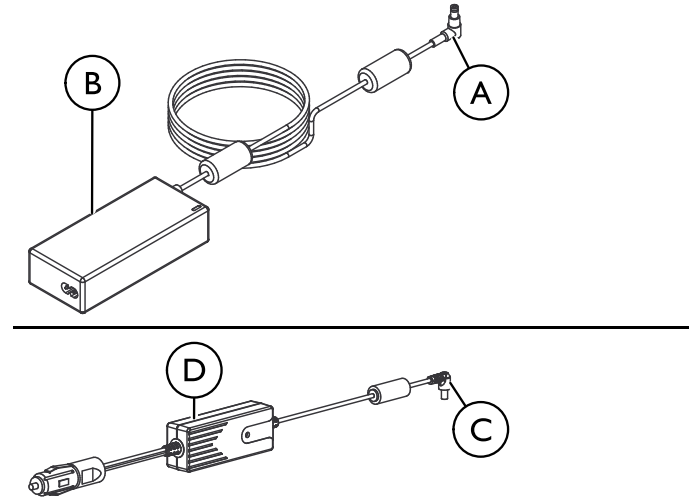
Make sure to check the internal battery is fully charged after charging the batteries and prior to using the XPO<sub>2</sub> concentrator for the first time and prior to any subsequent use.

The four battery gauge LEDs **A** light up when the internal battery is fully charged.

1. Turn On the XPO<sub>2</sub> concentrator by pressing and holding the On/Off button.
2. Press and hold the battery status button located on the keypad.

## 5.6 Charging the Internal Battery

1. To charge the internal battery, do one of the following:

- Connect the AC power adapter **A** (if charging from a wall outlet) to the input power connector located on the side of the XPO<sub>2</sub> concentrator and then plug the other end of the power adapter into the power source.
- Connect the DC power adapter **C** (if charging from a vehicle) to the input power connector located on the side of the XPO<sub>2</sub> concentrator and then plug the other end of the power adapter **D** into the power source.

If internal battery is completely discharged, it will fully charge in approximately 3 to 4 hours. The battery gauge will illuminate during charging. It is recommended to recharge the internal battery, even if only partially depleted, as often as possible.

## 5.7 Charging the Supplemental Battery

1. Connect the AC **A** or DC **B** power adapter to the supplemental battery pack **C** (Detail “A” and “B”).
2. Connect the other end of the AC **A** or DC **B** power adapter into the appropriate power source.

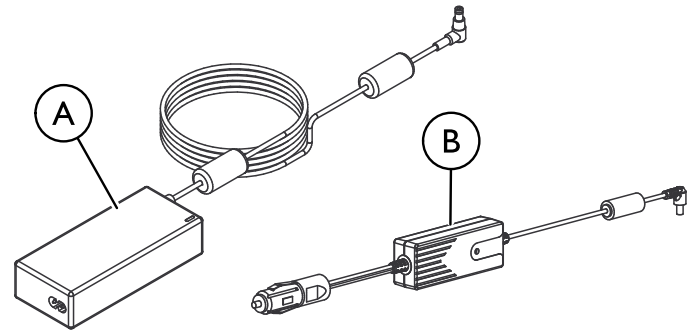
If completely discharged, the supplemental battery will fully charge in approximately 3 to 4 hours. Supplemental Battery Gauge indicator lights DO NOT illuminate during charging.

### Checking Charge Level of Supplemental Battery


Pressing the battery status button while charging turns the battery gauge on and shows the status of the battery capacity. The battery gauge also flashes “the last” or highest light bar to show that the battery pack is charging. The flashing bar shows the battery is actually being charged. The lit bars show the status of the battery capacity being restored.


Detail “A”

Detail “B”
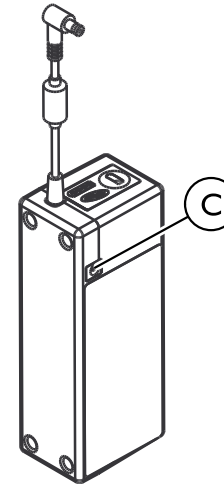
## 5.8 Charging the Internal and Supplemental Battery Pack Simultaneously

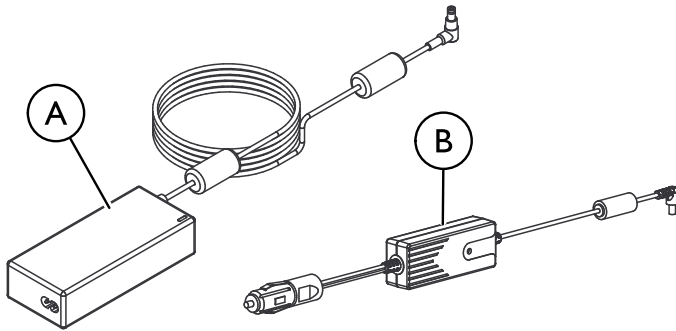- I. To charge the supplemental battery pack and internal battery together, perform the following:
  - a. Connect the AC (A) or DC power (B) to the input power connector of the supplemental battery pack (C) as shown.
  - b. Connect the supplemental battery pack to the input power connector located on the side of the XPO<sub>2</sub> unit (D) as shown.
  - c. Plug the other end of the adapter into the power source.

The internal battery gauge will illuminate during charging. Supplemental battery gauge indicator lights **DO NOT** illuminate during charging.

You may operate your XPO<sub>2</sub> concentrator while the internal and/or the supplemental battery is being charged. Use of the XPO<sub>2</sub> concentrator during charging will lengthen the amount of time required to reach full charge status of the batteries.

## 5.9 Battery Life and Management

Time away from home is greatly extended by combining the use of the AC power supply, DC power supply and the XPO<sub>2</sub> concentrator's batteries. To ensure the batteries maintain their optimal charge level, utilize the AC power supply whenever you have access to electric power. Utilize the DC power supply whenever you are in a vehicle.

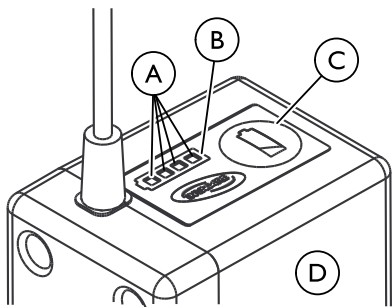
### WARNING!

#### Risk of Injury

Battery exhaustion will result in a loss of supplemental oxygen. To ensure proper supplemental oxygen delivery during a power outage:

- Plug your portable concentrator into a power source.

© = Battery Status Button

### Do's

- **When you first receive your XPO<sub>2</sub> concentrator, fully charge the internal battery (and supplemental external battery if provided) overnight.**

- Keep your batteries ④ fully charged when using the XPO<sub>2</sub> concentrator on a daily basis.
- Your XPO<sub>2</sub> concentrator and supplemental external battery can be recharged at any time when using the XPO<sub>2</sub> concentrator on a daily basis.
- Always ensure the XPO<sub>2</sub> concentrator internal and supplemental external batteries are recharged as soon as possible after they become fully discharged. The batteries may be permanently degraded if left fully discharged for an extended length of time.
- Check the status of your XPO<sub>2</sub> concentrator internal and supplemental external battery once a month if you are not using your XPO<sub>2</sub> concentrator on a daily basis. Both batteries should be maintained at 2 illuminated bars ⑤ worth of charge if not using the XPO<sub>2</sub> concentrator on a daily basis.
- Heat is the worst enemy of a battery. Allow plenty of air to circulate around the XPO<sub>2</sub> concentrator so that the battery is kept as cool as possible when charging and also when in use.

### Don'ts

- Do Not use or leave the XPO<sub>2</sub> concentrator or supplemental external battery in excessive heat or cold.
- Do Not store or leave the XPO<sub>2</sub> concentrator or supplemental external battery in car trunks, etc. for extended periods of time.
- Do Not store either battery fully charged (4 bars illuminated on the unit's battery gauge ⑥) if you are going to store your XPO<sub>2</sub> concentrator for any time greater than a day. Recharge or discharge the battery to 2 bars (50% charge) only. Storing a battery with a full charge may degrade its useful life.
- Do Not leave your supplemental external battery plugged into the XPO<sub>2</sub> concentrator when the XPO<sub>2</sub> concentrator is not in use. The supplemental external battery will lose charge while plugged into the XPO<sub>2</sub> concentrator even with the XPO<sub>2</sub> concentrator turned off.

## 6 Maintenance

### 6.1 Cleaning the Cabinet

#### **DANGER!**

#### **Risk of Injury or Damage**

Liquid will damage the internal components of the concentrator. To avoid damage or injury from electrical shock:

- Turn Off the XPO<sub>2</sub> concentrator and unplug the power cord before cleaning.
- DO NOT allow any cleaning agent to drip inside the air inlet and outlet openings.
- DO NOT spray or apply any cleaning agent directly to the cabinet.
- DO NOT hose down the product.

#### **CAUTION!**

#### **Risk of damage**

Harsh chemical agents can damage the concentrator. To avoid damage:

- DO NOT clean the cabinet, carry bag, or filter with alcohol and alcohol based products (isopropyl alcohol), concentrated chlorine-based products (ethylene chloride), and oil-based products (Pine-Sol<sup>®</sup>, Lestoil<sup>®</sup>) or any other harsh chemical agents. Only use mild liquid dish detergent (such as Dawn<sup>™</sup>).

Periodically clean the XPO<sub>2</sub> concentrator's cabinet as follows:

1. Remove the XPO<sub>2</sub> concentrator from the carrying bag.
2. Use a damp cloth, or sponge, with a mild detergent such as Dawn<sup>™</sup> dish washing soap to gently clean the exterior case.

3. Allow the concentrator to air dry, or use a dry towel, before returning the concentrator to the carry bag or, operating the concentrator.
4. Return the XPO<sub>2</sub> concentrator to its carry bag.

### 6.2 Cleaning the Air Intake Filter

#### **CAUTION!**

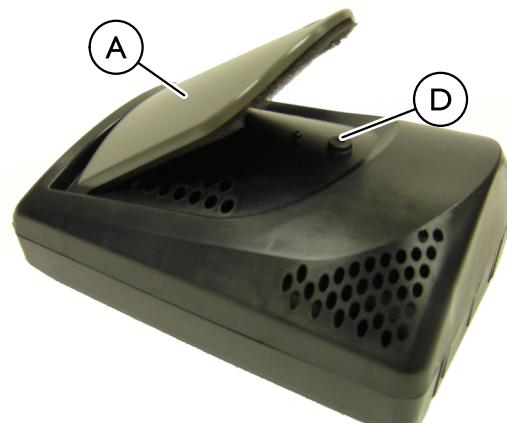
#### **Risk of damage**

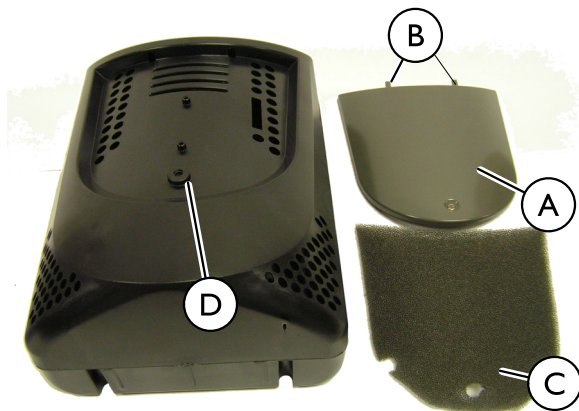
To avoid damage from clogging:

- DO NOT operate the XPO<sub>2</sub> concentrator without the air intake filter installed.

Remove the air intake filter and clean it at least once each week.

1. Lift filter cover (A) slightly and pull down to remove tabs (B) from the grooves.
2. Lift out the filter (C).
3. Use a vacuum cleaner or wash with a mild liquid dish detergent (such as Dawn™) and water. Rinse thoroughly.
4. Thoroughly dry the filter and inspect for fraying, crumbling, tears and holes. Replace filter if any is found.
5. Reinstall the filter cover by placing the tabs in the slots and engaging the magnet (D).

 Use only Invacare part number I157692 as the air intake filter for your XPO<sub>2</sub> concentrator.


### 6.3 Cleaning the Carrying Bag
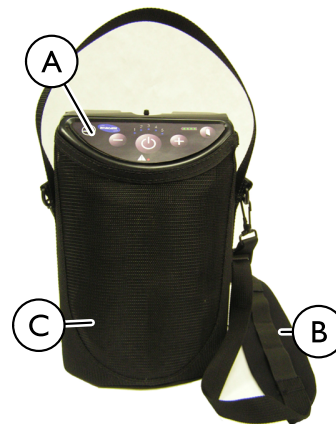
#### **CAUTION!**

#### **Risk of damage**

To avoid damage to the product:  
– DO NOT machine wash or dry the bag.

(B) = Shoulder strap

1. Remove the XPO<sub>2</sub> concentrator (A) from the bag (C).
2. Wipe or brush the carry bag with a mild liquid dish detergent (such as Dawn™) and water. Rinse thoroughly.
3. Allow the bag to air dry after cleaning and before using.

Use only Invacare carrying bag, model XPO150.

### 6.4 Maintenance Checklist

**Model No:** \_\_\_\_\_

**Serial No:** \_\_\_\_\_

<b>ON EACH INSPECTION</b>							
Record Date of Service							
Record Elapsed Hours on Hour Meter							
Clean Cabinet Filter(s)							
<b>EVERY 4,320 HRS, BETWEEN PATIENTS, OR ANNUALLY, WHICHEVER COMES FIRST</b>							
Check Oxygen Concentration*							
<b>AS NEEDED</b>							
Clean/Replace Cabinet Filter							
Check/Replace Outlet HEPA Filter*							
Check/Replace Compressor Inlet Filter*							

 4,320 hours are equivalent to usage 24 hours per day, 7 days per week for 6 months

\*To be conducted by provider or qualified service technician. Refer to service manual.

---

## 7 After Use

---

### 7.1 Storage

1. Store the repackaged oxygen concentrator in a dry area.
2. DO NOT place objects on top of packaged concentrator.

### 7.2 Disposal

Recycle

DO NOT dispose of in household waste

This product has been supplied from an environmentally aware manufacturer who complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2012/19/EC. This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

Follow local governing ordinances and recycling plans regarding disposal of the concentrator or components normally used in operation. The concentrator does not generate waste or residue in operation.

- DO NOT dispose of the concentrator in the normal waste stream.
- Any accessories not part of the concentrator **MUST** be handled in accordance with the individual product marking for disposal.
- DO NOT dispose of the internal or supplemental battery packs. Battery packs should be returned to your dealer/provider.

## 8 Troubleshooting

### 8.1 Troubleshooting

If your XPO<sub>2</sub> Portable Oxygen Concentrator fails to operate properly, refer to the following chart for possible causes and solutions. If necessary, contact your service provider.

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>START-UP INDICATOR</b>		
XPO <sub>2</sub> concentrator does not operate when On/Off button pressed. OR Front panel lights DO NOT illuminate upon start-up OR All front panel lights go dark.	Upon arrival, internal battery is shipped in shut down mode.	Connect XPO <sub>2</sub> concentrator to AC power adapter
	On/Off button was not held down long enough.	Try to power up the unit again while continuing to press the On/Off button until the front panel lights begin to illuminate. This takes typically 2-3 seconds.
	Internal battery is discharged (and supplemental battery if connected to the unit).	Connect XPO <sub>2</sub> concentrator to either the AC or DC power adapter and retry.
	Malfunction	Change to an alternate oxygen supply and contact your equipment provider.
<b>NO BREATH DETECTED ALARM:</b>		
Continuous audible beep and red alarm indicator is on continuously (not blinking) upon start up.	XPO <sub>2</sub> concentrator has not detected a breath for a predetermined time period (60 seconds).	1. Verify the cannula is connected, not kinked, properly positioned and you are breathing through your nose. 2. If the alarm continues, change to another source of oxygen and contact your equipment provider.

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>CHARGING INDICATOR:</b>		
With the concentrator turned Off and with the AC or DC charging adapter plugged into the concentrator, no Battery Gauge segments are illuminated when battery status button is pressed. Battery Gauge is dark.	Power source is no good, or there is a loose connection.	Try another power outlet and check connections to charger and concentrator.
	Internal Battery is outside the allowed temperature range for charging.	Allow unit to reach temperature range for charging (between 41°F and 95°F).
	For DC adapter only, blown fuse.	Change fuse. Use Invacare part number I185420 as fuse for DC adapter.
<b>LOW BATTERY WARNING:</b>		
There is a single audible beep and the green 25% battery indicator is blinking.	Internal battery requires charging.	Connect XPO <sub>2</sub> concentrator to its AC or DC power adapter, or to its supplemental battery pack.
<b>LOW BATTERY ALARM:</b>		
There is a single audible beep and the green 25% battery indicator is rapidly blinking.	Internal battery requires charging.	<ol style="list-style-type: none"> <li>1. Connect XPO<sub>2</sub> concentrator to its AC or DC power adapter, or to its supplemental battery pack.</li> <li>2. If no other power source is available, change to a reserve oxygen supply.</li> </ol>
<b>BATTERY DISCHARGED ALARM:</b>		
There is a triple audible beep and the green 25% battery indicator is blinking very rapidly.	Internal battery is completely discharged and needs to be recharged.	<ol style="list-style-type: none"> <li>1. Connect XPO<sub>2</sub> concentrator to its AC or DC power adapter, or to its supplemental battery pack.</li> <li>2. If no other power source is available, change to a reserve oxygen supply.</li> </ol>

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>BREATH RATE OVER CAPACITY ALARM:</b>		
<p>Intermittent audible beep and illuminated red alarm indicator.</p> <p>Audible beeps and red indicator blinks become more rapid as breathing rate increases.</p>	<p>Your breathing rate has exceeded the capacity of the XPO<sub>2</sub> concentrator.</p>	<ol style="list-style-type: none"> <li>1. Immediately reduce your activity level to get your breath rate to slow down, and/or</li> <li>2. If the alarm continues, change to another source of oxygen.</li> </ol>
<b>SYSTEM TOO HOT/COLD TO START ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated. Fan is On.</p> <p>AND</p> <p>Flow indicators 1 &amp; 2 are illuminated.</p>	<p>Concentrator is too hot, or too cold, to allow it to turn On.</p>	<ol style="list-style-type: none"> <li>1. Allow concentrator to cool down to less than 95°F (35°C), or warm up to 50°F (10°C), before turning the concentrator Off, then On again.</li> <li>2. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>
<b>SYSTEM RUNNING TOO HOT/COLD ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated. Fan is On.</p> <p>AND</p> <p>Flow indicators 1 &amp; 3 are illuminated.</p>	<p>Concentrator has become too hot, or too cold, during operation.</p>	<ol style="list-style-type: none"> <li>1. Allow concentrator to cool down to less than 95°F (35°C), or warm up to 50°F (10°C), before turning the concentrator On again.</li> <li>2. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>BATTERY TOO HOT/COLD ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated.</p> <p>AND</p> <p>Flow indicators 1 &amp; 4 are illuminated.</p>	<p>Internal Battery has become too hot, or too cold, during operation.</p>	<ol style="list-style-type: none"> <li>1. Unplug chargers if connected.</li> <li>2. Allow concentrator to cool down to less than 95°F (35°C), or warm up to 50°F (10°C), before resuming charging, or turning Off the concentrator and turning the concentrator On again.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>
<b>STUCK BUTTON ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated.</p> <p>AND</p> <p>Flow indicators 1 &amp; 5 are illuminated.</p>	<p>A stuck button is being detected upon power up.</p>	<ol style="list-style-type: none"> <li>1. Turn Off the concentrator.</li> <li>2. With the concentrator turned Off, press each control button looking for a stuck button.</li> <li>3. Turn On the concentrator while ensuring that you are not continuing to hold the Power On/Off button once the lights begin to illuminate.</li> <li>4. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>OPERATING ALARM:</b>		
Continuous audible beep and red alarm indicator is illuminated. AND Flow indicators 3 & 4 are illuminated.	Abnormal compressor operation has been detected.	<ol style="list-style-type: none"> <li>1. Turn Off the concentrator.</li> <li>2. Turn On the concentrator.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>
<b>COMPRESSOR ALARM:</b>		
Continuous audible beep and red alarm indicator is illuminated. AND Flow indicators 4 & 5 are illuminated.	Abnormal operation has been detected.	<ol style="list-style-type: none"> <li>1. Turn Off the concentrator.</li> <li>2. Turn On the concentrator.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>
<b>SYSTEM ALARM:</b>		
Continuous audible beep and red alarm indicator is illuminated. AND Flow indicators 3, 4 & 5 are illuminated.	Abnormal system operation has been detected.	<ol style="list-style-type: none"> <li>1. Turn Off the concentrator.</li> <li>2. Turn On the concentrator.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>

## **9 Warranty**

---

### **9.1 Limited Warranty - Europe and Australia**

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located inside the back cover of this manual.

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Invacare Deutschland GmbH