# Operator's Manual

## Invacare<sup>®</sup> XPO<sub>2</sub><sup>™</sup> Portable Concentrator

English UK .....	2
Italian .....	46
Spanish .....	98
Portuguese .....	148
Greek .....	196
Dutch .....	253

**Dealer:** This manual MUST be given to the end user.

**User:** BEFORE using this product, read this manual and save for future reference.

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

**Yes, you can.<sup>®</sup>**

---

**⚠ WARNING**

**DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.**

---

**⚠ ACCESSORIES WARNING**

**INVACARE PRODUCTS ARE SPECIFICALLY DESIGNED AND MANUFACTURED FOR USE IN CONJUNCTION WITH INVACARE ACCESSORIES. ACCESSORIES DESIGNED BY OTHER MANUFACTURERS HAVE NOT BEEN TESTED BY INVACARE AND ARE NOT RECOMMENDED FOR USE WITH INVACARE PRODUCTS.**

---

---

# TABLE OF CONTENTS

<b>SPECIAL NOTES</b> .....	<b>4</b>
Disposal of Equipment and Accessories .....	5
<b>SECTION 1—GENERAL GUIDELINES</b> .....	<b>6</b>
Operating Information .....	7
Maintenance .....	8
Radio Frequency Interference .....	9
Polarized Plug Instruction .....	10
<b>SECTION 2—PACKAGING AND HANDLING</b> .....	<b>11</b>
Unpacking .....	11
Inspection .....	11
Storage .....	11
<b>SECTION 3—TECHNICAL DESCRIPTION</b> .....	<b>12</b>
Intended Use .....	12
Technical Description .....	12
<b>SECTION 4—XPO<sub>2</sub> FEATURES</b> .....	<b>13</b>
<b>SECTION 5—TYPICAL PRODUCT PARAMETERS</b> .....	<b>14</b>
Regulatory Listing .....	16
<b>SECTION 6—OPERATING INSTRUCTIONS</b> .....	<b>17</b>
Introduction .....	17
Filters .....	17
Location .....	18
Recommendations for Optimal Performance .....	18
Power Supplies .....	19
Checking Charge Level of Internal Battery .....	21
Charging the Batteries .....	22
Battery Time Management .....	26
Prolonging Battery Life .....	26
Powering the XPO <sub>2</sub> .....	28
Connecting/Positioning the Nasal Cannula .....	30
Adjusting the Pulse Flow Setting .....	31
Using the XPO <sub>2</sub> Portable Concentrator .....	32
Indicators, Warnings and Alarms .....	33
<b>SECTION 7—CLEANING, CARE, AND MAINTENANCE</b> .	<b>37</b>
Cabinet .....	37
Air Intake Filter .....	38
Carrying Bag .....	39
<b>SECTION 8—TROUBLESHOOTING GUIDE</b> .....	<b>40</b>
<b>SECTION 9—OPTIONAL ACCESSORIES</b> .....	<b>45</b>
<b>CUSTOMER SERVICE AND WARRANTY INFORMATION</b> .....	<b>302</b>

# SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the following table for definitions of the signal words.

SIGNAL WORD	MEANING
 DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
 WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

---

## NOTICE

The information contained in this document is subject to change without notice.

---

## DANGER

**DO NOT SMOKE** while using this device. Keep all matches, lit cigarettes, candles or other sources of ignition out of the room in which this product is located and away from where oxygen is being delivered.

**NO SMOKING** signs should be prominently displayed. Textiles and other materials that normally would not burn are easily ignited and burn with great intensity in oxygen enriched air. Failure to observe this warning can result in severe fire, property damage and cause physical injury or death.

---

---

## Disposal of Equipment and Accessories

Follow local governing ordinances and recycling plans regarding disposal of device components. **DO NOT** dispose of the internal or supplemental battery packs. Battery packs should be returned to your dealer/provider.

Recycle

**DO NOT** dispose of  
in household waste

---

### **⚠ CAUTION**

“Caution: Statutory law can restrict this device to sale or rental by or on order of a physician, or any other practitioner licensed by the law of the government agency in which he/she practices to use or order the use of this device.”

Invacare recommends an alternate source of supplemental oxygen in the event of a power outage, alarm condition or mechanical failure. Consult your physician or equipment provider for the type of reserve system required.

This equipment is to be used as an oxygen supplement and is not considered life supporting or life sustaining.

---

## SECTION I—GENERAL GUIDELINES

In order to ensure the safe installation, assembly and operation of the XPO<sub>2</sub> Portable Concentrator these instructions **MUST** be followed.

---

### **⚠ WARNING**

This section contains important information for the safe operation and use of this product.

---

### **⚠ DANGER**

**Risk of electric shock. DO NOT disassemble. Refer servicing to qualified service personnel. No user serviceable parts.**

**TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS**

**A spontaneous and violent ignition may occur if oil, grease, greasy substances, or petroleum based products come in contact with oxygen under pressure. These substances **MUST** be kept away from the XPO<sub>2</sub> portable concentrator, tubing and connections, and all other oxygen equipment. **DO NOT** use any lubricants unless recommended by Invacare.**

**Avoid using while bathing. If continuous usage is required by the physician's prescription, the concentrator **MUST** be located in another room at least 7 ft (2.1 m) from the bath.**

**DO NOT come in contact with the concentrator while wet.**

**DO NOT place or store product where it can drop into water or other liquid.**

**DO NOT reach for product that has fallen into water. Unplug **IMMEDIATELY**.**

---

**⚠ DANGER**

Keep the oxygen tubing, cord, and unit out from under such items as blankets, bed coverings, chair cushions, clothing and away from heated or hot surfaces, including space heaters, stoves and similar electrical appliances.

Avoid creation of any spark near medical oxygen equipment. This includes sparks from static electricity created by any type of friction.

---

DO NOT move or relocate concentrator by pulling on the cord.

A product should NEVER be left unattended when plugged in. Make sure XPO<sub>2</sub> is Off when not in use.

Outdoor use of the XPO<sub>2</sub> MUST be conducted with internal or supplemental battery power only.

## Operating Information

The XPO<sub>2</sub> cannot be used in conjunction with PAP, Bi-Level or other such devices.

If the XPO<sub>2</sub> is not working properly, if it has been dropped or damaged, or dropped into water, call equipment provider/qualified technician for examination and repair.

If you feel ill or uncomfortable, or if the unit does not signal an oxygen pulse and you are unable to hear and/or feel the oxygen pulse, consult your equipment provider and/or your physician IMMEDIATELY.

NEVER drop or insert any object or liquid into any opening.

DO NOT use extension cords with AC power adaptors provided.

For optimum performance, Invacare recommends that each concentrator be on and running for a minimum of 5 minutes. Shorter periods of operation may reduce maximum product life.

The XPO<sub>2</sub> is not designed to be used with a humidifier. Use of this device with a humidifier may impair performance and/or damage the equipment.

Use of any means or device to reduce fire propagation in the supply accessories is contraindicated and may cause improper oxygen therapy delivery.

When your automobile is turned Off, disconnect the car accessory power supply and remove the XPO<sub>2</sub> from the automobile. NEVER allow the XPO<sub>2</sub> to be stored in a very hot or cold automobile or in other similar, high or low, temperature environments. Refer to Typical Product Parameters on page 14.

DO NOT operate in temperatures below 41° F (5°C) or above 104° F (40° C) for extended periods of time.

The XPO<sub>2</sub> MUST be used in an upright position.

Invacare recommends the XPO<sub>2</sub> not be used in the rain.

## Maintenance

The XPO<sub>2</sub> was specifically designed to minimize routine preventive maintenance. Only professionals of the healthcare field or persons fully conversant with this process such as factory trained personnel should perform preventive maintenance or performance adjustments on the oxygen concentrator.

## Radio Frequency Interference

This equipment has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2. These limits are designed to provide a reasonable protection against electromagnetic interference in a typical medical installation.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standards. To determine if the emissions from the XPO<sub>2</sub> are causing the interference, turn the XPO<sub>2</sub> Off. If the interference with the other device(s) stops, then the XPO<sub>2</sub> is causing the interference. In such rare cases, interference may be reduced or corrected by one of the following measures:

- Reposition, relocate, or increase the separation between the equipment.
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

DO NOT connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices.

Close supervision is necessary when this product is used near children or physically-challenged individuals.

Additional monitoring or attention may be required for patients using this device who are unable to hear or see alarms or communicate discomfort.

Be aware that electrical cords and/or tubing could present a tripping hazard.

A change in altitude may affect total oxygen available to you. Consult your physician before traveling to higher or lower altitudes to determine if your flow settings should be changed.

## **Polarized Plug Instruction**

As a safety feature, this appliance may have a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. **DO NOT** attempt to defeat this safety feature.

## **SECTION 2—PACKAGING AND HANDLING**

### **Unpacking**

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the carrier, or your local dealer.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton. The Invacare XPO<sub>2</sub> Portable Oxygen Concentrator packaging contains the following items (as shown below). If any parts are missing, please contact your equipment provider.
  - XPO<sub>2</sub> portable oxygen concentrator with carrying bag
  - Operator's manual
  - AC power adapter
  - DC power adapter
  - Supplemental battery (Model XPO100B only)
  - Language Overlay Labels

*NOTE: Retain all containers and packing materials for storage or return shipment.*

### **Inspection**

Inspect/examine exterior of the oxygen concentrator and accessories for damage. Inspect all components.

### **Storage**

1. Store the repackaged oxygen concentrator in a dry area.
2. DO NOT place objects on top of packaged concentrator.

## **SECTION 3—TECHNICAL DESCRIPTION**

### **Intended Use**

The Invacare portable concentrator is to be used by patients with respiratory disorders who require supplemental oxygen. The device is not intended to sustain or support life.

### **Technical Description**

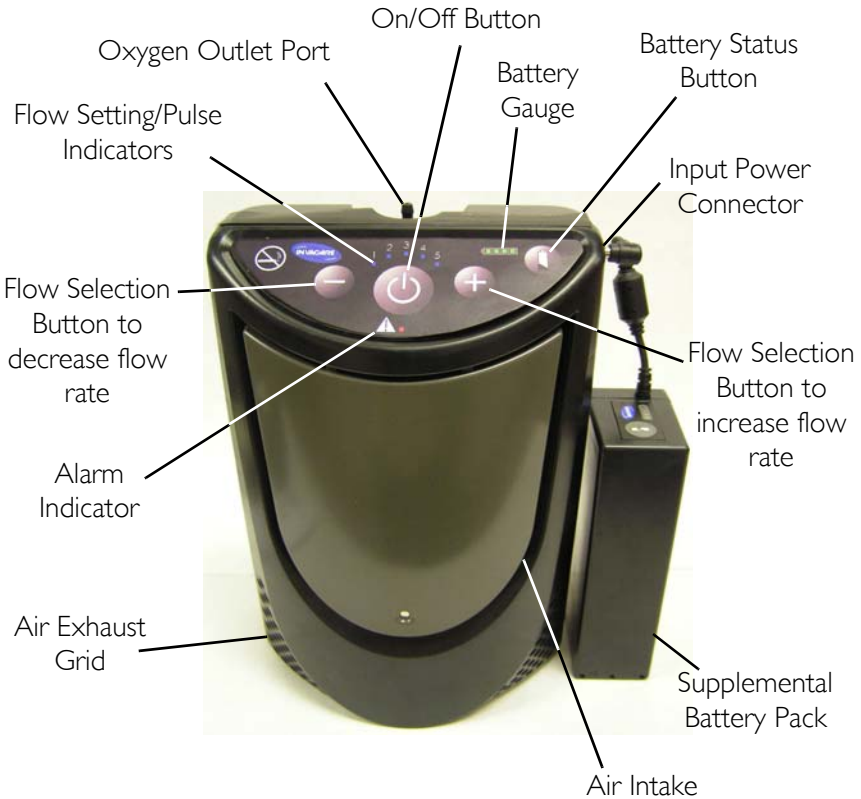
The oxygen concentration level of the output gas ranges from 87% to 95.6%. The oxygen is delivered to the patient through the use of a nasal cannula. When the demand for oxygen is detected, the oxygen is delivered through pulsed flow with pulse flow settings of 1 through 5.

The Invacare portable concentrator uses a molecular sieve and pressure swing adsorption methodology to produce the oxygen gas output. Ambient air enters the device, is filtered and then compressed. This compressed air is then directed toward one of two nitrogen adsorbing sieve beds. Concentrated oxygen exits the opposite end of the active sieve bed and is directed into an oxygen reservoir where it is delivered to the patient in specific volumes during the inhalation portion of a detected breath.

The Invacare portable concentrator is capable of operation by the patient in a home environment, in an institutional environment or in a vehicle or other mobile environment. Device standard power options include an AC to DC switching power supply operating from AC power outlet (120 VAC/ 60 Hertz or 230VAC/50 Hertz nominal), a DC to DC switching power supply operating from accessory outlets typically found in a mobile vehicle type environment (12 VDC nominal) and a supplemental rechargeable battery.

## SECTION 4—XPO<sub>2</sub> FEATURES

Please review the figures below to familiarize yourself with the locations of the XPO<sub>2</sub> features and controls.

**FIGURE 4.1** XPO<sub>2</sub> Features

# SECTION 5—TYPICAL PRODUCT PARAMETERS

	Direct Current
	Type BF equipment
	Attention - Consider Accompanying Documents
	DO NOT smoke
	Class II, Double Insulation
	Power On/Off
	DO NOT dispose of in household waste
	Recycle
	DO NOT use oil or grease
	Keep dry
<b>IPX1</b>	Protected against dripping water
	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Electrical Requirements:	AC Power Supply: 230 VAC, 50 Hertz, 0.7 Amps
	DC Power Supply: 11 VDC, 6.5 Amps
Rated Current Input:	3.8 Amps at 18 VDC
Sound Level @ setting 2:	≤ 45dBA weighted

## SECTION 5—TYPICAL PRODUCT PARAMETERS

Altitude:	Up to 10,000 ft (3046 m) above sea level
Oxygen Concentration:*	87% to 95.6%, after initial warm-up period (approximately 5 minutes)
*Based on an atmospheric pressure of 14.7 psi (101 kPa) at 70°F (21°C)	
Conservar Trigger Sensitivity: Factory set - no adjustment Pressure activated	≤ 0.20 cmH <sub>2</sub> O pressure drop  All settings - using 7 ft (2.1 m) cannula
Conservar Trigger Delay: Factory set - no adjustment	50 mSec  Nominal value - using 7 ft (2.1 m) cannula
Conservar Bolus Delivery: Fixed minute volume	Setting 1: 320 cc Setting 2: 480 cc Setting 3: 670 cc Setting 4: 800 cc Setting 5: 900 cc  Total cc's ± 75 cc delivered per minute
Conservar Breath Rate Capacity:	Up to, and including, 35 BPM without reduction of bolus minute volume
Dimensions:	10 in high x 7 in wide x 4 in deep (25.4 cm high x 17.8 cm wide x 10.2 cm wide)
Weight:	6.0 lbs (2.7 kg); 1.3 lbs (0.68 kg) for supplemental battery
Battery Duration (times are approximate):	Setting 1 = 3.5 hrs
	Setting 2 = 2.5 hrs
	Setting 3 = 2.0 hrs
	Setting 4 = 1.5 hrs
	Setting 5 = 1.0 hrs
Battery Recharge Time: <i>NOTE: Recharge time increases if battery is charging while unit is running.</i>	4 hours

## SECTION 5—TYPICAL PRODUCT PARAMETERS

Humidity	Operating Humidity: 15% to 60% non condensing
	Storage Humidity: up to 95% non condensing
Temperature Range: (All power sources)	Operating temperature: 41°F to 95°F (5°C to 35°C)
	Storage temperature: -2°F to 140°F (-20°C to 60°C)
Extended Temperature Range: (Using AC or DC adapters)	Operating Temperature: 95°F to 104° (35°C to 40°C) Continuous use - all settings
Extended Temperature Range (Using Internal Battery)	Operating Temperature: 95°F to 104° (35°C to 40°C) Continuous use - settings 1, 2 and 3 45 minutes (max) - setting 4 30 minutes (max) - setting 5

## Regulatory Listing

ETL certified complying with:	EN 55011: 1998
	CISPR 11: 2003
	IEC 60601-1: 1995
	IEC 60601-1-2: 2.1 Ed.
	IEC 61000-3-2: 2005
	IEC 61000-3-3: 2005
	EN 60601-1-2: 2nd Ed.
	EN 61000-3-2: 2000
	EN 61000-3-3: 1995
	UL 60601-1, 1st ed.
	CSA 601.1 M90
	93/42/EEC Class IIa

# SECTION 6—OPERATING INSTRUCTIONS

## Introduction

Oxygen concentrators were introduced in the mid-1970's to provide supplemental oxygen to those in need. Oxygen concentrators separate nitrogen from air to create a high concentration oxygen gas output. Until now, the size and weight of traditional oxygen concentrator's have limited their portability. The XPO<sub>2</sub> delivers a pulse flow of oxygen in a lightweight package that can be easily carried and used away from home.

This operator's manual will acquaint you with the XPO<sub>2</sub> and its accessories. Please make sure you read and understand all the information contained in this manual before you operate your unit. Should you have any questions, please contact your equipment provider.

## Filters

Air enters the XPO<sub>2</sub> through an air intake filter located under the cover on the front of the unit. This filter prevents hair and other large particles in the air from entering the unit. Before you operate the XPO<sub>2</sub>, make sure this filter is clean, dry and properly positioned.

To clean/replace the air intake filter, refer to Cleaning, Care, And Maintenance on page 37.

## Location

### **⚠ WARNING**

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like.

Keep unit at least 3 in (7.6 cm) away from walls, draperies, furniture, and the like.

Locate and position the XPO<sub>2</sub> in a well ventilated space so that the air intake and the air exhausts are not obstructed.

## Recommendations for Optimal Performance

Operating Temperature:	41°F to 95°F (5°C to 35°C)
Relative Humidity:	15% to 60%
Transport/Storage Temperature:	-2°F to 140°F (-20°C to 60°C) Allow unit to warm, or cool, to operating temperature range before using.
Electrical:	No extension cords.
Altitude:	Up to 10,000 ft (3046 m) above sea level.
Tubing and Cannula:	4 ft (1.2 m) cannula with a maximum 25 ft (7.6 m) of crush resistant tubing (DO NOT pinch).
Environment:	Smoke, pollutant and fume free. No confined spaces (example: no closets).
Time of Operation:	Up to 24 hours per day when connected to AC or DC power adapter.

---

## Power Supplies

---

### **⚠ WARNING**

Use only Invacare specified power supplies with the XPO<sub>2</sub> portable concentrator. Use of other non approved power supplies with the XPO<sub>2</sub> can cause damage and/or injury and will void the warranty.

---

*NOTE: For this procedure, refer to FIGURE 6.1.*

The XPO<sub>2</sub> can be supplied power from a factory installed internal battery, from an AC power adapter connected to the wall outlet, from a DC power adapter connected to a DC outlet like that found in an automobile, or from a supplemental battery pack.

**Internal Battery:** A factory installed, rechargeable, internal battery is located inside the XPO<sub>2</sub>. When fully charged, it supplies power for up to 3.5 hours. An audible alarm sounds when the battery power is getting low. Refer to Indicators, Warnings and Alarms on page 33.

**AC Power Adapter:** An AC power adapter allows the XPO<sub>2</sub> to be connected to a 100-240 volt 50/60 hertz outlet. The power adapter converts AC voltage to a DC voltage that can be used to power the XPO<sub>2</sub>. Use of the AC power adapter will allow the XPO<sub>2</sub> to be operated and simultaneously recharge the internal battery. This AC power adapter can also be used to recharge the supplemental battery pack. Refer to Charging the Batteries on page 22.

**DC Power Adapter:** A DC power adapter allows the XPO<sub>2</sub> to be connected to an automobile's (boat, motor home, etc....) 12-volt DC outlet. Use of the DC power adapter will allow the XPO<sub>2</sub> to be operated and simultaneously recharge the internal battery. This DC power adapter can also be used to recharge the supplemental battery pack. Refer to Charging the Batteries on page 22.

**Supplemental Battery Pack:** The XPO<sub>2</sub> can be powered by a supplemental battery pack. This battery pack can be attached to the XPO<sub>2</sub> carrying bag and connected to the power input connector of the unit. The supplemental battery pack will provide up to 3.5 hours of additional run time. The supplemental battery pack has its own battery status button and battery gauge to identify the level of battery charge.

*NOTE: The supplemental battery cannot be used to charge the internal battery.*

**FIGURE 6.1** Power Supplies


## Checking Charge Level of Internal Battery

*NOTE: For this procedure, refer to FIGURE 6.2.*

*NOTE: Perform this procedure after charging the batteries and prior to using the XPO<sub>2</sub> for the first time and prior to any subsequent use to ensure the internal battery is fully charged.*

1. Turn On the XPO<sub>2</sub> by pressing and holding the On/Off button.
2. Press and hold the battery status button located on the keypad.

*NOTE: Pressing the battery status button will illuminate the battery gauge and indicate the charge level of the internal battery. When the battery is fully charged all four battery gauge indicator lights will illuminate and stay illuminated until the button is released. Lesser charges will illuminate fewer gauge indicator lights.*

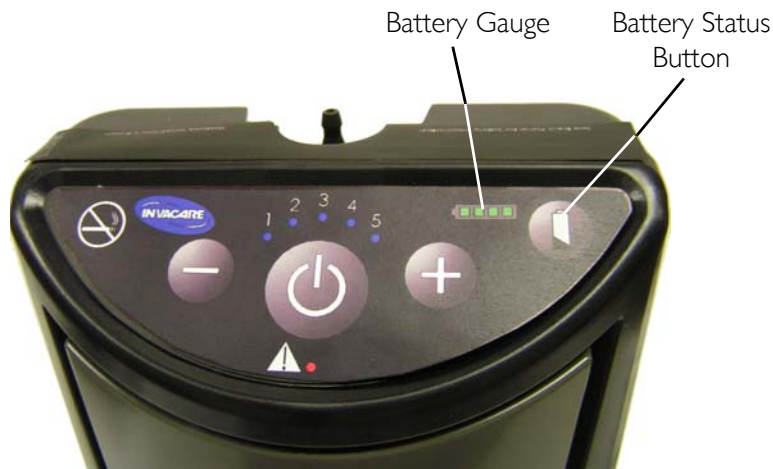**FIGURE 6.2** Checking Charge Level of Internal Battery


## Charging the Batteries

### Charging the Internal Battery

*NOTE: For this procedure, refer to FIGURE 6.3.*

1. To charge the internal battery, do one of the following:
  - Connect the AC power adapter (if charging from a wall outlet) to the input power connector located on the side of the unit and then plug the other end of the power adapter into the power source.
  - Connect the DC power adapter (if charging from a vehicle) to the input power connector located on the side of the unit and then plug the other end of the power adapter into the power source.

*NOTE: If internal battery is completely discharged, it will fully charge in approximately 3 to 4 hours. The battery gauge will illuminate during charging. It is recommended to recharge the internal battery, even if only partially depleted, as often as possible.*

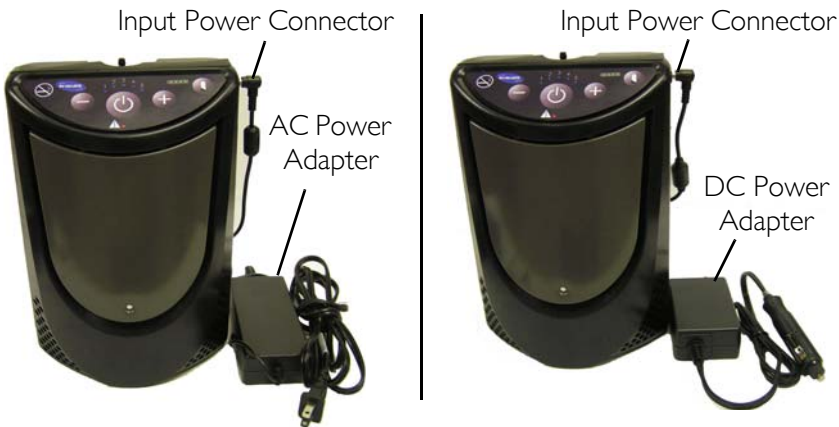**FIGURE 6.3** Charging the Internal Battery

## **Charging the Supplemental Battery Only**

*NOTE: For this procedure, refer to FIGURE 6.4 on page 24.*

1. Connect the AC (Detail “A”) or DC (Detail “B”) power adapter to the supplemental battery pack.
2. Connect the other end of the AC (Detail “A”) or DC (Detail “B”) power adapter into the appropriate power source.

*NOTE: If completely discharged, the supplemental battery will fully charge in approximately 3 to 4 hours. Supplemental Battery Gauge indicator lights DO NOT illuminate during charging.*

## **Supplemental Battery Status Button**

*NOTE: For this procedure, refer to Detail “C” in FIGURE 6.4 on page 24.*

*NOTE: Pressing the battery status button while charging turns the battery gauge On and shows the status of the battery capacity. The battery gauge also flashes “the last” or highest light bar to show that the battery pack is charging. The flashing bar shows the battery is actually being charged. The lit bars show the status of the battery capacity being restored.*

**DETAIL “A” -  
USING AC POWER  
ADAPTER**

**DETAIL “B” -  
USING DC POWER  
ADAPTER**

**DETAIL “C”**

**FIGURE 6.4** Charging the Supplemental Battery Only

## Charging the Internal and Supplemental Battery Pack Simultaneously

*NOTE: For this procedure, refer to FIGURE 6.5.*

1. To charge the supplemental battery pack and internal battery together, perform the following:
  - A. Connect the AC or DC power adapter to the supplemental battery pack as shown.
  - B. Connect the supplemental battery pack to the XPO<sub>2</sub> as shown.
  - C. Plug the other end of the adapter into the power source.

*NOTE: The internal battery gauge will illuminate during charging. Supplemental battery gauge indicator lights DO NOT illuminate during charging.*

*NOTE: You may operate your XPO<sub>2</sub> Portable Concentrator while the internal and/or the supplemental battery is being charged. Use of the XPO<sub>2</sub> during charging will lengthen the amount of time required to reach full charge status of the batteries.*

**FIGURE 6.5** Charging the Internal and Supplemental Battery Pack Simultaneously

## Battery Time Management

Time away from home is almost limitless by combining the use of the AC power supply, DC power supply and the XPO<sub>2</sub> batteries. To ensure the batteries maintain their optimal charge level, utilize the AC power supply whenever you have access to electric power. Utilize the DC power supply whenever you are in a vehicle.


## Prolonging Battery Life

### Do's

- When you first receive your XPO<sub>2</sub>, fully charge the internal battery (and supplemental external battery if provided) overnight.
- Keep your batteries fully charged when using the XPO<sub>2</sub> regularly.
- Your XPO<sub>2</sub> and supplemental external battery can be recharged at any time. These batteries can have their charge “topped off” at any time.
- Always ensure the XPO<sub>2</sub> internal and supplemental external batteries are recharged as soon as possible after they become fully discharged. The batteries may be permanently degraded if left fully discharged for an extended length of time.
- Check the status of your XPO<sub>2</sub> internal and supplemental external battery once a month if you are not using your XPO<sub>2</sub> regularly. Both batteries should be maintained at 2 illuminated bars worth of charge.
- Heat is the worst enemy of a battery. Allow plenty of air to circulate around the XPO<sub>2</sub> so that the battery is kept as cool as possible when charging and also when in use.

## Don'ts

- DO NOT use or leave the XPO<sub>2</sub> or supplemental external battery in excessive heat or cold.
- DO NOT store or leave the XPO<sub>2</sub> or supplemental external battery in car trunks, etc. for extended periods of time.
- DO NOT store either battery fully charged (4 bars illuminated on the unit's battery gauge) if you are going to store your XPO<sub>2</sub> for any time greater than 2 weeks. Recharge or discharge the battery to 2 bars (50% charge) only. Storing a battery with a full charge may degrade its useful life.
- DO NOT leave your supplemental external battery plugged into the XPO<sub>2</sub> when the XPO<sub>2</sub> is not in use. The supplemental external battery will lose charge while plugged into the XPO<sub>2</sub> even with the XPO<sub>2</sub> turned off.

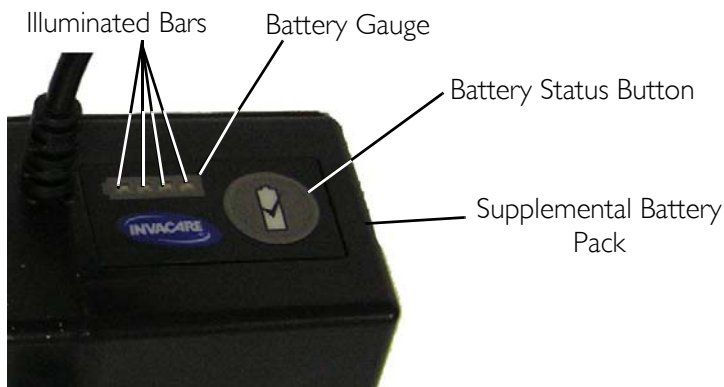**FIGURE 6.6** Prolonging Battery Life

## Powering the XPO<sub>2</sub>

*NOTE: Both the internal and supplemental battery require full charging prior to first use. Refer to Charging the Batteries on page 22.*

*NOTE: For this procedure, refer to FIGURE 6.7 on page 29.*

1. The XPO<sub>2</sub> allows the freedom to choose from the following power sources:
  - The internal battery. The XPO<sub>2</sub> comes equipped with an internal battery that is installed at the factory and is not user serviceable.
  - The supplemental battery pack. To use, connect the supplemental battery pack to the XPO<sub>2</sub> as shown in Detail “A”.
  - A DC power outlet (i.e. an automobile). To use, connect the DC power adapter to the XPO<sub>2</sub> as shown in Detail “B”. Connect the other end of the DC power adapter to the DC power source.
  - An AC outlet (i.e. wall outlet of your home). To use, connect the AC power adapter to the XPO<sub>2</sub> as shown in Detail “C”. Connect the other end of the AC power adapter to the AC power outlet.
2. If the XPO<sub>2</sub> is in use and connection to an external power supply is necessary, perform the following:
  - A. Turn off the XPO<sub>2</sub>.
  - B. Connect the external power supply (e.g. supplemental battery pack, DC mobile power adapter, or AC power adapter) to the XPO<sub>2</sub>.
  - C. Turn the XPO<sub>2</sub> on.

**DETAIL “A” - SUPPLEMENTAL BATTERY PACK**

**DETAIL “B” - DC POWER ADAPTER**

**DETAIL “C” - AC POWER ADAPTER**

**FIGURE 6.7** Powering the XPO2

## Connecting/Positioning the Nasal Cannula

---

### **⚠ CAUTION**

To ensure proper breath detection and oxygen delivery, DO NOT use tubing length exceeding 25 ft (7.6 m).

---


*NOTE: For this procedure, refer to FIGURE 6.8.*

*NOTE: Invacare recommends using the XPO<sub>2</sub> with a 4 ft (1.2 m) cannula. The tubing must be connected to the XPO<sub>2</sub> oxygen outlet as shown below.*

*NOTE: Replace the nasal cannula on a regular basis. Check with your equipment provider or physician to determine how often the cannula should be replaced.*

*NOTE: DO NOT share cannulas between patients.*

1. Connect the nasal cannula to the XPO<sub>2</sub> oxygen outlet port (Detail "A").
2. Place the cannula over your ears and position the prongs in your nose as instructed by your health care provider or cannula manufacturer (Detail "B").

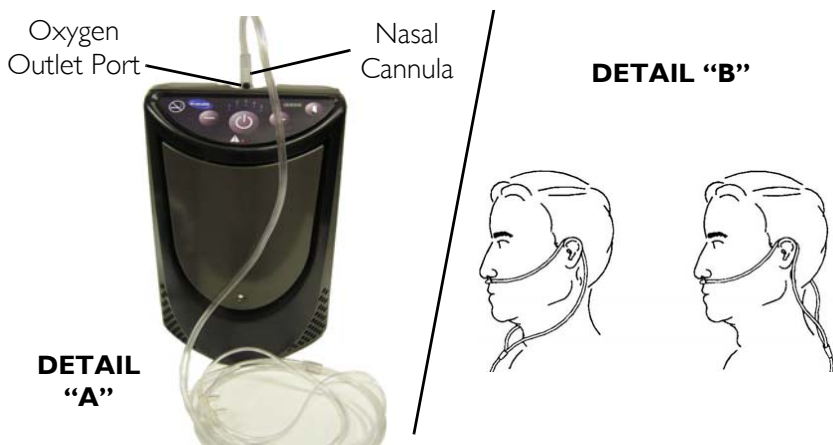**FIGURE 6.8** Connecting/Positioning the Nasal Cannula

## Adjusting the Pulse Flow Setting

### **⚠ CAUTION**

It is very important to select the prescribed level of oxygen flow. DO NOT increase or decrease the flow unless a change has been prescribed by your physician or therapist.

*NOTE: For this procedure, refer to FIGURE 6.9 on page 32.*

1. Turn On the XPO<sub>2</sub> by pressing and holding the power On/Off button for approximately one second. Each time the XPO<sub>2</sub> is turned On, a brief alarm will sound. The XPO<sub>2</sub> pulse flow setting at power-up will be the same flow setting selected the last time the unit was turned Off.
2. The XPO<sub>2</sub> has five pulse flow settings. Press the flow selection (+ or -) button until your prescribed flow setting (1 to 5) is illuminated by a blue indicator light.

FLOW SETTING	AVERAGE PULSE VOLUME (AT 20 BREATHS PER MINUTE)
1	15 ml

2	23 ml
3	31 ml
4	37 ml
5	42 ml

**FIGURE 6.9** Adjusting the Pulse Flow Setting

## Using the XPO<sub>2</sub> Portable Concentrator

1. Turn the XPO<sub>2</sub> On by pressing the On/Off button.
2. Breathe normally through your nose. Breathing through your mouth may result in less effective oxygen therapy. The XPO<sub>2</sub> will put out a measured pulse of oxygen each time it detects inhalation. The blue flow setting indicator light will blink each time an inhalation is detected and the XPO<sub>2</sub> outputs a pulse of oxygen.
3. Turn the XPO<sub>2</sub> Off by pressing and holding the power On/Off button for approximately two seconds.

*NOTE: The time required for the XPO<sub>2</sub> to reach maximum oxygen concentration after being turned On is approximately 5 minutes.*

## Indicators, Warnings and Alarms

Start-Up Indicator – The XPO<sub>2</sub> will turn On all lights briefly and check for stuck buttons upon power up. It will then output a brief audible beep when first turned On to indicate the unit is operational.

Breath Detect Indicator - The XPO<sub>2</sub> will put out a measured pulse of oxygen each time it detects an inhalation. The blue flow setting indicator light will blink each time an inhalation is detected and the XPO<sub>2</sub> outputs a pulse of oxygen.

Internal Battery Charging Indicator – When the unit is not turned On, but the AC, or DC, charger is connected to the unit and providing power, the Battery Gauge Display will sequentially illuminate each of the four segments. Periodically the sequential illumination will be interrupted by the actual battery charge display. If the unit is fully charged, all four segments will be illuminated continuously until the unit is turned On, or removed from charger power. If no Battery Gauge Display segments are illuminated, the unit is not charging. The lack of charging could be due to either a loss of power, loose charger connections or the internal battery is not at the proper temperature for charging. Allow the unit to warm, or cool, to recommended charging temperature range (see Recommendations for Optimal Performance on page 18), check for loose connections and/or try a different power outlet.

Low Battery Warning – When the XPO<sub>2</sub> internal battery (or supplemental battery, if applicable) capacity drops to 25%, a brief audible beep will sound. The green 25% battery gauge indicator will intermittently blink.

Low Battery Alarm – When the XPO<sub>2</sub> internal battery (or supplemental battery, if applicable) capacity drops to 15%, a double audible beep will sound. The green 25% battery gauge indicator will rapidly blink.

Battery Discharged Alarm – Once the XPO<sub>2</sub> internal battery (or supplemental battery, if applicable) capacity drops to minimum, a triple audible beep will sound and the GREEN 25% battery gauge indicator will blink very rapidly. The unit will then shutdown.

No Breath Detect Alarm - When the XPO<sub>2</sub> is operating but does not sense breathing, a constant audible alarm sounds and the red alarm indicator light will illuminate continuously. If this occurs, check the connection from the cannula to the XPO<sub>2</sub> unit and ensure that you are breathing through your nose. If the alarm continues, change to an alternate source of oxygen and contact your equipment provider.

Breath Rate Over Capacity Alarm – If your breathing rate causes the capacity of the XPO<sub>2</sub> to be exceeded, an intermittent audible beep sounds and the red alarm indicator will illuminate intermittently. When this occurs, reduce your activity and/or change to another source of oxygen.

If your breathing rate continues to increase the audible beeps and red indicator blinks will become more rapid.

*NOTE: Oxygen will be supplied during this alarm.*

System Too Hot/Cold for Start Alarm – If the internal sensors register temperatures outside factory set levels upon start-up, the unit will alarm with a rapid audible beep, the red alarm indicator light will illuminate continuously, the unit will not operate and the flow setting 1 & 2 blue indicator lights will illuminate. The fan will turn On. Allow the unit to cool or warm to the recommended operating temperature range (see Recommendations for Optimal Performance on page 18). Turn Off the unit and try again. Change to an alternate source of oxygen while waiting. If the alarm continues, contact your equipment provider.

System Too Hot/Cold Running Alarm – If the internal sensors register temperatures outside factory set levels during operation, the unit will alarm with a rapid audible beep, the red alarm indicator light will illuminate continuously, the unit will not operate and the flow setting **1 & 3** blue indicator lights will illuminate. The fan will turn On. Allow the unit to cool or warm to the recommended operating temperature range (see Recommendations for Optimal Performance on page 18). Turn Off the unit and try again. Change to an alternate source of oxygen while waiting. If the alarm continues, contact your equipment provider.

Battery Too Hot/Cold Alarm – If the internal battery sensor registers temperature outside a factory defined temperature range while the unit is operating, the unit will alarm with a rapid audible beep, the red alarm indicator light will illuminate continuously, the unit will stop running and the flow setting **1 & 4** blue indicator lights will illuminate. Turn Off the unit and disconnect AC or DC power adapters. Allow to the unit to cool or warm, to the recommended operating temperature range (see Recommendations for Optimal Performance on page 18) and try again. Change to an alternate source of oxygen while waiting. If the alarm continues, contact your equipment provider.

Stuck Button Alarm – During power up of the unit, if it detects that a button is stuck or being pressed too early, the unit will alarm with a rapid audible beep, the red alarm indicator light will illuminate continuously, the unit will not operate and the flow setting **1 & 5** blue indicator lights will illuminate. Turn Off the unit. Individually press all the front panel buttons, looking for buttons that seem stuck in a “pressed” position. Try turning On the unit again without pressing any other buttons during the power up. If the alarm continues, change to an alternate source of oxygen and contact your equipment provider.

Operating Alarm – If unit detects abnormal operating conditions in the unit, the unit will alarm with a rapid audible beep, the red alarm indicator light will illuminate continuously, the unit will not operate and the flow setting **3 & 4** blue indicator lights will illuminate. Turn Off and then turn On the unit. If the alarm continues, you should change to another source of oxygen and contact your equipment provider.

Compressor Alarm – If unit detects abnormal compressor conditions, the unit will alarm with a rapid audible beep, the red alarm indicator light will illuminate continuously, the unit will not operate and the flow setting **4 & 5** blue indicator lights will illuminate. Turn Off and then turn On the unit. If the alarm continues, you should change to another source of oxygen and contact your equipment provider.

System Alarm – If unit detects abnormal system conditions, the unit will alarm with a rapid audible beep, the red alarm indicator light will illuminate continuously, the unit will not operate and the flow setting **3, 4 & 5** blue indicator lights will illuminate. Turn Off and then turn On the unit. If the alarm continues, you should change to another source of oxygen and contact your equipment provider.

Audible Alarms - To temporarily silence the audible alarms, quickly press the On/Off button. Except for the system Too Hot/Too Cold alarms, this feature will reset itself, every 30 seconds, each time a new alarm is triggered, or if the unit is turned Off. When performed during the system Too Hot/Too Cold alarms, the audible alarm will be silenced until a new alarm is triggered, or the unit is turned Off.

## **SECTION 7—CLEANING, CARE, AND MAINTENANCE**

---

### **⚠ WARNING**

Turn Off the XPO<sub>2</sub> and unplug the power cord before cleaning.

DO NOT allow any cleaning agent to drip inside the air inlet and outlet openings.

DO NOT spray or apply any cleaning agent directly to the cabinet.

---

### **⚠ CAUTION**

DO NOT clean the cabinet, carry bag, or filter with alcohol and alcohol based products (isopropyl alcohol), concentrated chlorine-based products (ethylene chloride), and oil-based products (Pine-Sol<sup>®</sup>, Lestoil<sup>®</sup>) or any other harsh chemical agents. Only use mild liquid dish detergent (such as Dawn<sup>™</sup>).

---

## **Cabinet**

Periodically clean the XPO<sub>2</sub> cabinet as follows:

1. Remove the XPO<sub>2</sub> from the carrying bag.
2. Use a damp cloth, or sponge, with a mild detergent such as Dawn dish washing soap to gently clean the exterior case.
3. Allow the unit to air dry, or use a dry towel, before returning the unit to the carry bag or, operating the unit.
4. Return the XPO<sub>2</sub> to its carry bag.

## Air Intake Filter

### **⚠ CAUTION**


DO NOT operate the XPO<sub>2</sub> without the air intake filter installed.

*NOTE: For this procedure, refer to FIGURE 7.1.*

*NOTE: Remove the air intake filter and clean it at least once each week.*

1. Lift filter cover slightly and pull down to remove tabs from the grooves.
2. Lift out the filter.
3. Use a vacuum cleaner or wash with a mild liquid dish detergent (such as Dawn™) and water. Rinse thoroughly.
4. Thoroughly dry the filter and inspect for fraying, crumbling, tears and holes. Replace filter if any is found.
5. Reinstall the filter cover by placing the tabs in the slots and engaging the magnet.

*NOTE: Use only Invacare part number 1148281 as the air intake filter for your XPO<sub>2</sub>.*

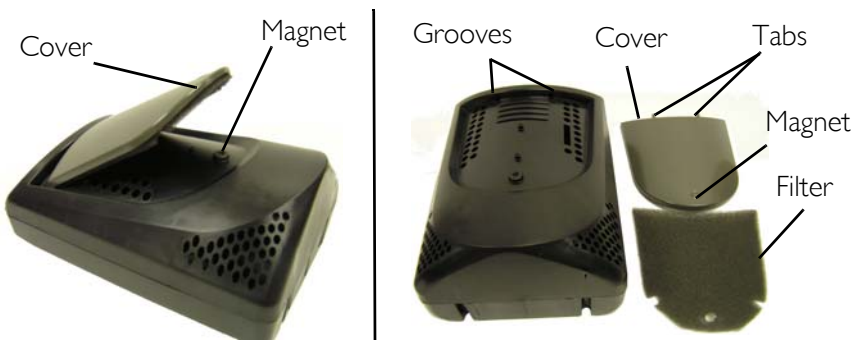**FIGURE 7.1** Air Intake Filter

## Carrying Bag

*NOTE: For this procedure, refer to FIGURE 7.2.*

---

### **⚠ CAUTION**

DO NOT machine wash or dry the bag.

---

1. Remove the XPO<sub>2</sub> from the bag
2. Wipe or brush the carry bag with a mild liquid dish detergent (such as Dawn™) and water. Rinse thoroughly.
3. Allow the bag to air dry after cleaning and before using.

*NOTE: Use only Invacare carrying bag, model XPO150.*

**FIGURE 7.2** Carrying Bag

## SECTION 8—TROUBLESHOOTING GUIDE

If your XPO<sub>2</sub> Portable Oxygen Concentrator fails to operate properly, refer to the following chart for possible causes and solutions. If necessary, contact your service provider.

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>START-UP INDICATOR</b>		
XPO <sub>2</sub> does not operate when On/Off button pressed.  OR  Front panel lights DO NOT illuminate upon start-up  OR  All front panel lights go dark.	On/Off button was not held down long enough.  Internal battery is discharged (and supplemental battery if connected to the unit).  Malfunction	Try to power up the unit again while continuing to press the On/Off button until the front panel lights begin to illuminate. This takes typically 2-3 seconds.  Connect XPO <sub>2</sub> to either the AC or DC charger and retry.  Change to an alternate oxygen supply and contact your equipment provider.
<b>NO BREATH DETECTED ALARM:</b>		
Continuous audible beep and red alarm indicator is on continuously (not blinking) upon start up.	XPO <sub>2</sub> has not detected a breath for a predetermined time period.	1. Verify the cannula is connected, not kinked, properly positioned and you are breathing through your nose. 2. If the alarm continues, change to another source of oxygen and contact your equipment provider.

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>CHARGING INDICATOR</b>		
With the unit turned Off and with the AC or DC charging adapter plugged into the unit, no Battery Gauge segments are illuminated. Battery Gauge is dark.	Power source is no good, or there is a loose connection.	Try another power outlet and check connections to charger and unit.
	Internal Battery is outside the allowed temperature range for charging.	Allow unit to cool down to less than 95° F (35° C), or warm up to 50° F (10° C).
<b>LOW BATTERY WARNING:</b>		
There is a single audible beep and the green 25% battery indicator is blinking.	Internal battery requires charging.	Connect XPO <sub>2</sub> to its AC or DC power adapter, or to its supplemental battery pack.
<b>LOW BATTERY ALARM:</b>		
There is a double audible beep and the green 25% battery indicator is rapidly blinking.	Internal battery requires charging.	<ol style="list-style-type: none"> <li>1. Connect XPO<sub>2</sub> to its AC or DC power adapter, or to its supplemental battery pack.</li> <li>2. If no other power source is available, change to a reserve oxygen supply.</li> </ol>
<b>BATTERY DISCHARGED ALARM:</b>		
There is a triple audible beep and the green 25% battery indicator is blinking very rapidly.	Internal battery is completely discharged and needs to be recharged.	<ol style="list-style-type: none"> <li>1. Connect XPO<sub>2</sub> to its AC or DC power adapter, or to its supplemental battery pack.</li> <li>2. If no other power source is available, change to a reserve oxygen supply.</li> </ol>

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>BREATH RATE OVER CAPACITY ALARM:</b>		
<p>Intermittent audible beep and illuminated red alarm indicator.</p> <p>Audible beeps and red indicator blinks become more rapid as breathing rate increases.</p>	<p>Your breathing rate has exceeded the capacity of the XPO<sub>2</sub>.</p>	<ol style="list-style-type: none"> <li>1. Immediately reduce your activity level to get your breath rate to slow down, and / or</li> <li>2. If the alarm continues, change to another source of oxygen.</li> </ol>
<b>SYSTEM TOO HOT/COLD TO START ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated. Fan is On.</p> <p>AND</p> <p>Flow indicators 1 &amp; 2 are illuminated.</p>	<p>Unit is too hot, or too cold, to allow it to turn On.</p>	<ol style="list-style-type: none"> <li>1. Allow unit to cool down to less than 95° F (35° C), or warm up to 50° F (10° C), before turning the unit Off, then On again.</li> <li>2. If the alarm continues, change to another source of oxygen and contact your equipment provider</li> </ol>
<b>SYSTEM RUNNING TOO HOT/COLD ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated. Fan is On.</p> <p>AND</p> <p>Flow indicators 1 &amp; 3 are illuminated.</p>	<p>Unit has become too hot, or too cold, during operation.</p>	<ol style="list-style-type: none"> <li>1. Allow unit to cool down to less than 95° F (35° C), or warm up to 50° F (10° C), before turning the unit On again</li> <li>2. If the alarm continues, change to another source of oxygen and contact your equipment provider</li> </ol>

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>BATTERY TOO HOT/COLD ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated.</p> <p>AND</p> <p>Flow indicators 1 &amp; 4 are illuminated.</p>	<p>Internal Battery has become too hot, or too cold, during operation.</p>	<ol style="list-style-type: none"> <li>1. Unplug chargers if connected.</li> <li>2. Allow unit to cool down to less than 95° F (35° C), or warm up to 50° F (10° C), before resuming charging or turning Off the unit and turning the unit On again.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider.</li> </ol>
<b>STUCK BUTTON ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated.</p> <p>AND</p> <p>Flow indicators 1 &amp; 5 are illuminated.</p>	<p>A stuck button is being detected upon power up.</p>	<ol style="list-style-type: none"> <li>1. Turn Off the unit</li> <li>2. With the unit turned Off, press each control button looking for a stuck button</li> <li>3. Turn On the unit while ensuring that you are not continuing to hold the Power On/Off button once the lights begin to illuminate.</li> <li>4. If the alarm continues, change to another source of oxygen and contact your equipment provider</li> </ol>

<b>ALARM/WARNING:</b>		
<b>SYMPTOM</b>	<b>PROBABLE CAUSE</b>	<b>SOLUTION</b>
<b>OPERATING ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated.</p> <p>AND</p> <p>Flow indicators 3 &amp; 4 are illuminated.</p>	<p>Abnormal operation has been detected.</p>	<ol style="list-style-type: none"> <li>1. Turn Off the unit.</li> <li>2. Turn On the unit.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider</li> </ol>
<b>COMPRESSOR ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated.</p> <p>AND</p> <p>Flow indicators 4 &amp; 5 are illuminated.</p>	<p>Abnormal compressor operation has been detected.</p>	<ol style="list-style-type: none"> <li>1. Turn Off the unit.</li> <li>2. Turn On the unit.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider</li> </ol>
<b>SYSTEM ALARM:</b>		
<p>Continuous audible beep and red alarm indicator is illuminated.</p> <p>AND</p> <p>Flow indicators 3, 4 &amp; 5 are illuminated.</p>	<p>Abnormal system operation has been detected.</p>	<ol style="list-style-type: none"> <li>1. Turn Off the unit.</li> <li>2. Turn On the unit.</li> <li>3. If the alarm continues, change to another source of oxygen and contact your equipment provider</li> </ol>

## **SECTION 9—OPTIONAL ACCESSORIES**

The following optional accessories and replacement parts (as shown below) are also available:

- Supplemental battery (provides up to additional 2.5 hours on Setting 2 of extended battery life), model number XPO110
- Carrying Bag, model number XPO150
- Wheeled cart with handle, model number XPO120
- Air intake filter, part number 1148281
- AC power adapter, power cord style:
  - N. America, Japan - model number XPO130
  - Europe - model number XPO132
  - Australia, New Zealand - model number XPO134
  - UK, Hong Kong, Vietnam - model number XPO 136
- DC Mobile power adapter, model number XPO140
- Accessory Bag XPO160

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**European Distributor Organisation:**

Invacare, Kleiststraße 49, D-32457 Porta Westfalica • Tel: (49) (0)57 31 754 540 •  
Fax: (49) (0)57 31 754 541 • [edo@invacare.com](mailto:edo@invacare.com) • [www.invacare.eu.com](http://www.invacare.eu.com)

**España:**

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona) •  
Tel: (34) (0)972 49 32 00 • Fax: (34) (0)972 49 32 20 • [contactsp@invacare.com](mailto:contactsp@invacare.com) •  
[www.invacare.es](http://www.invacare.es)

**France:**

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes • Tel: (33) (0)2 47 62 64 66 •  
Fax: (33) (0)2 47 42 12 24 • [contactfr@invacare.com](mailto:contactfr@invacare.com) • [www.invacare.fr](http://www.invacare.fr)

**Ireland:**

Invacare Ireland Ltd, Unit 5 Seatown Business Campus • Seatown Road, Swords,  
County Dublin – Ireland • Tel: (353) 1 810 7084 • Fax: (353) 1 810 7085 •  
[ireland@invacare.com](mailto:ireland@invacare.com) • [www.invacare.ie](http://www.invacare.ie)

**Italia:**

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI) • Tel: (39) 0445 38 00 59 •  
Fax: (39) 0445 38 00 34 • [italia@invacare.com](mailto:italia@invacare.com) • [www.invacare.it](http://www.invacare.it)

**Nederland:**

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede • Tel: (31) (0)318 695 757 •  
Fax: (31) (0)318 695 758 • [nederland@invacare.com](mailto:nederland@invacare.com) • [csede@invacare.com](mailto:csede@invacare.com) •  
[www.invacare.nl](http://www.invacare.nl)

**Norge:**

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo •  
Tel: (47) (0)22 57 95 00 • Fax: (47) (0)22 57 95 01 • [norway@invacare.com](mailto:norway@invacare.com) •  
[island@invacare.com](mailto:island@invacare.com) • [www.invacare.no](http://www.invacare.no)

**Österreich:**

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee •  
Tel: (43) 6232 5535 0 • Fax: (43) 6232 5535 4 • [info@invacare-austria.com](mailto:info@invacare-austria.com) •  
[www.invacare.at](http://www.invacare.at)

**Sverige & Suomi:**

Invacare AB, Fagerstagatan 9, S-163 91 Spånga • Tel: (46) (0)8 761 70 90 •  
Fax: (46) (0)8 761 81 08 • [sweden@invacare.com](mailto:sweden@invacare.com) • [finland@invacare.com](mailto:finland@invacare.com) •  
[www.invacare.se](http://www.invacare.se)

**Switzerland:**

Invacare AG, • Benkenstrasse 260, CH-4108 Witterswil • Tel: (41) (0)61 487 70 80 •  
Fax: (41) (0)61 487 70 81 • [switzerland@invacare.com](mailto:switzerland@invacare.com) • [www.invacare.ch](http://www.invacare.ch)

**United Kingdom:**

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ •  
Tel: (44) (0) 1656 776222 • Fax: (44) (0) 1656 776220 • [UK@invacare.com](mailto:UK@invacare.com) •  
[www.invacare.co.uk](http://www.invacare.co.uk)

**Portugal:**

Invacare Lda • Rua Estrada Velha, 949, P-4465-784 Leça do Balio •  
Tel: (351) (0)225 1059 46/47 • Fax: (351) (0)225 1057 39 • [portugal@invacare.com](mailto:portugal@invacare.com) •  
[www.invacare.pt](http://www.invacare.pt)

*Yes, you can:*

**Invacare Corporation**

[www.invacare.com](http://www.invacare.com)

**USA**

One Invacare Way  
Elyria, Ohio USA  
44036-2125  
440-329-6000  
800-333-6900  
Technical Services  
440-329-6593  
800-832-4707

EU Representative  
Invacare Deutschland  
GmbH  
Kleiststraße 49  
D-32457 Porta Westfalica  
Tel: (49) (0) 5731 754 0  
Fax: (49) (0) 5731 754  
52191

Invacare Corporation  
2101 E. Lake Mary Blvd.  
Sanford, FL 32773  
407-321-5630

© 2013 Invacare Corporation.  
All rights reserved. Republication,  
duplication or modification in whole or in  
part is prohibited without prior written  
permission from Invacare. Trademarks  
are identified by <sup>™</sup> and <sup>®</sup>. All trademarks  
are owned by or licensed to Invacare  
Corporation or its subsidiaries unless  
otherwise noted.

Pine-sol and Lestoil are trademarks of The  
Clorox Company.

Dawn is a trademark of The Proctor and  
Gamble Company.

Part No 1160887

Rev D - 10/14/13